Cortrium Apex 1.27

1.27 Apex Cortrium

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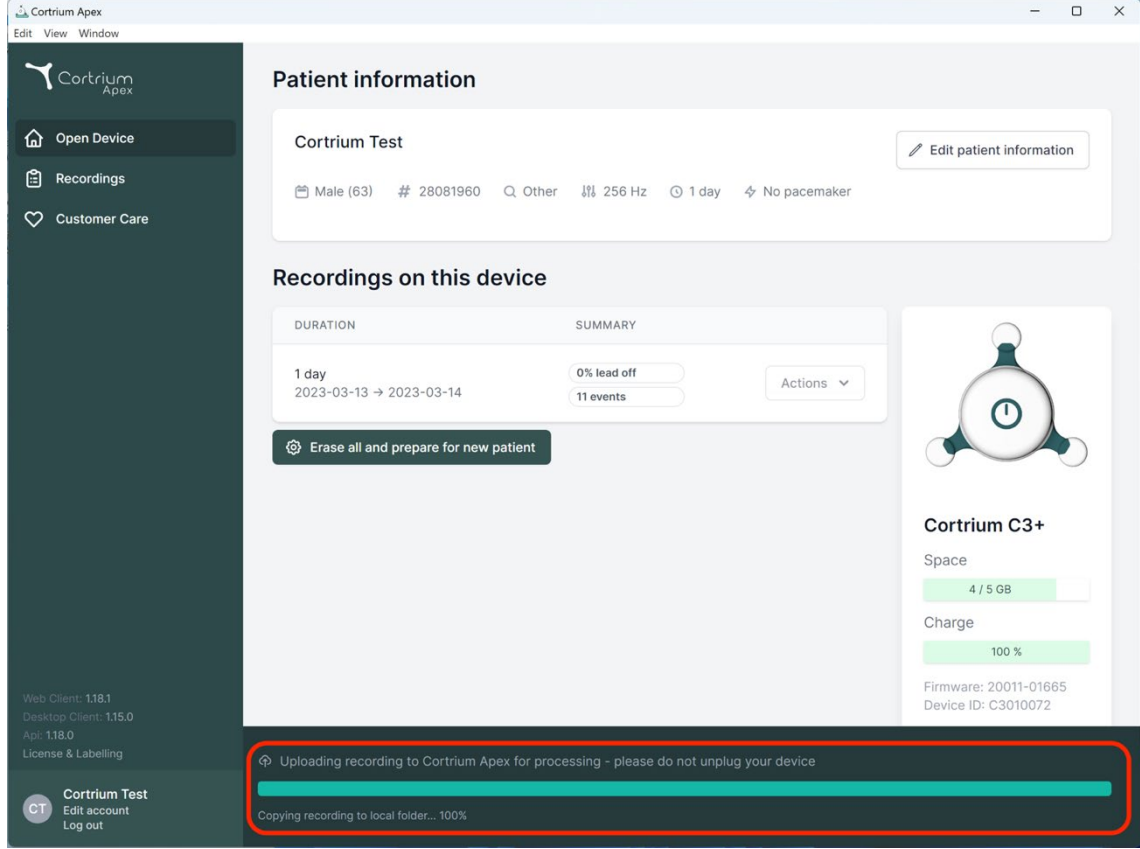
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and other parameters. Below this, the 'Recordings on this device' section shows a table with columns for 'DURATION' and 'SUMMARY'. A recording for '1 day' (2023-03-13 to 2023-03-14) is listed with '0% lead off' and '11 events'. To the right of the table is a device status card for 'Cortrium C3+' showing 'Space' (4 / 5 GB) and 'Charge' (100 %). At the bottom, a red-bordered box highlights a progress bar and status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'.

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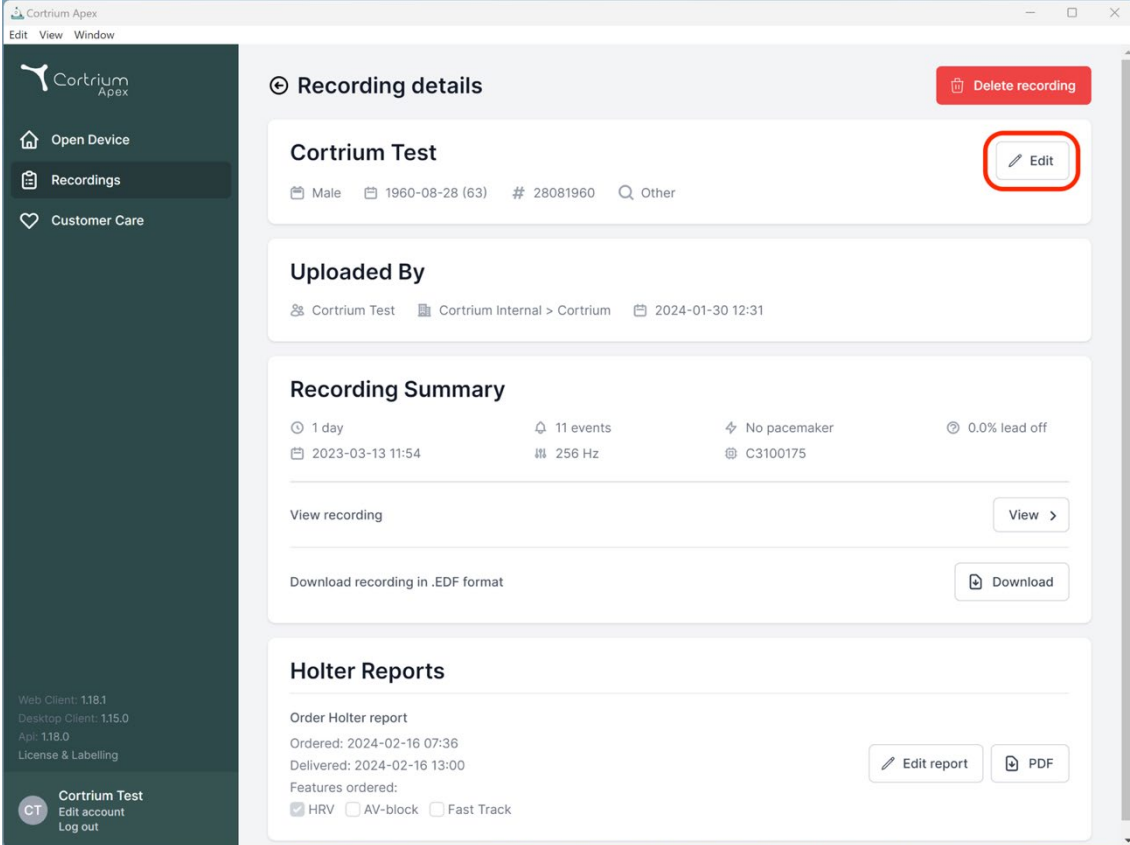
Recording details [Delete recording](#)

Cortrium Test [Edit](#)

Male 1960-08-28 (63) # 28081960 Q Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

[View recording](#) [View >](#)

[Download recording in .EDF format](#) [Download](#)

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Cancel Save

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
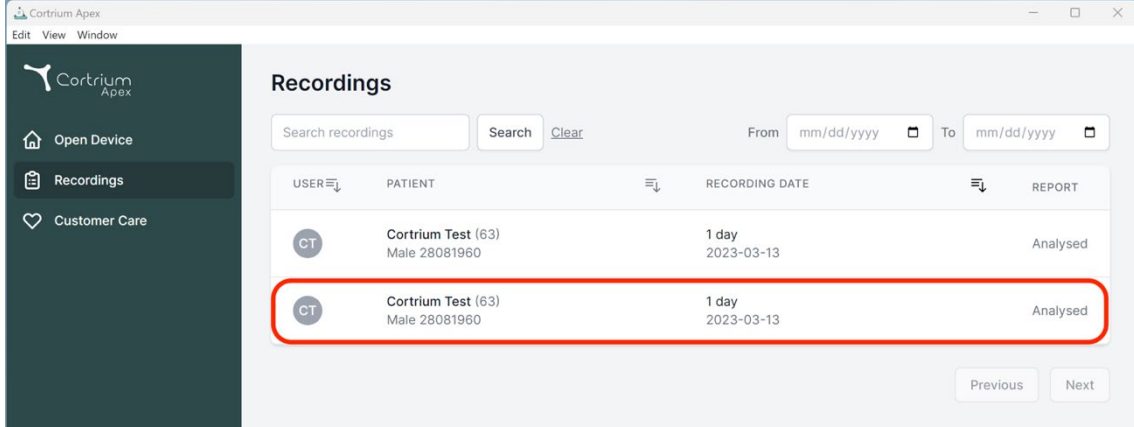
Recordings

Search recordings Search [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

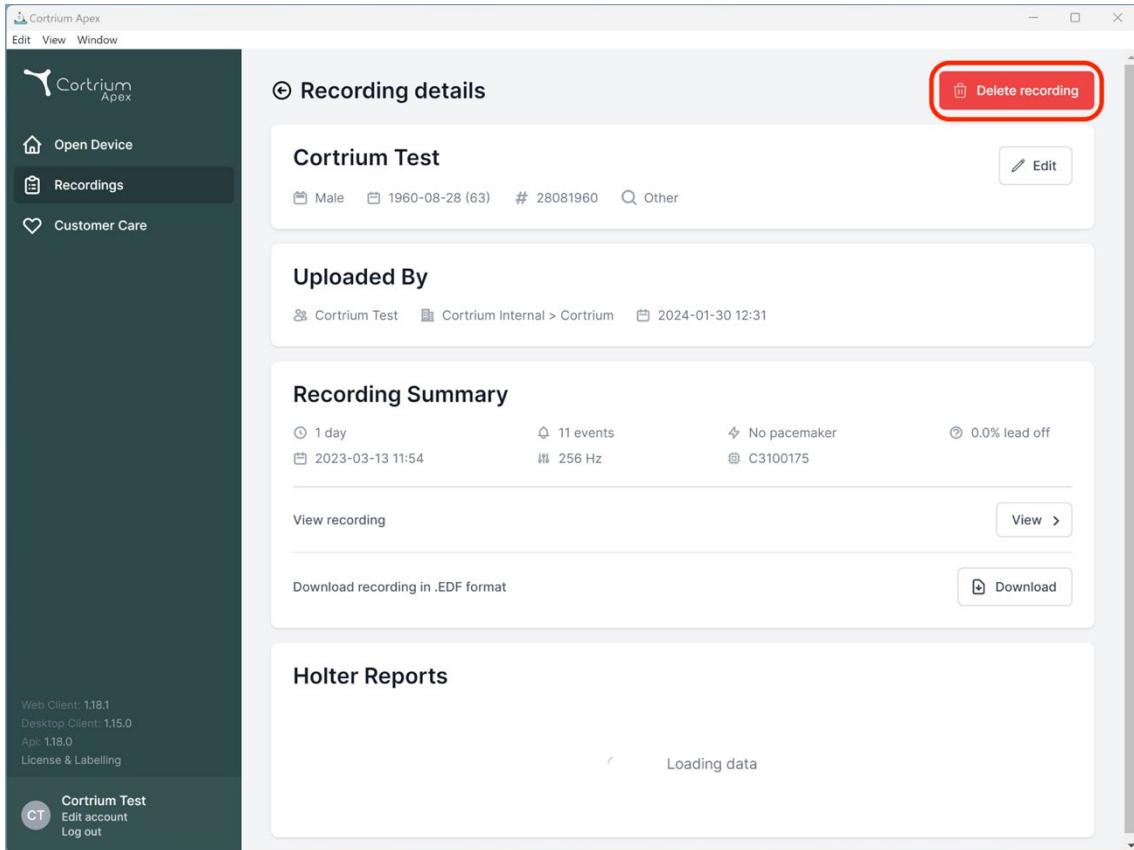Previous Next

Recording details [Delete recording](#)

Cortrium Test [Edit](#)

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording [View >](#)

Download recording in .EDF format [Download](#)

Holter Reports

Loading data

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

0.0% lead off

Download recording in .EDF format Download

Holter Reports

Loading data

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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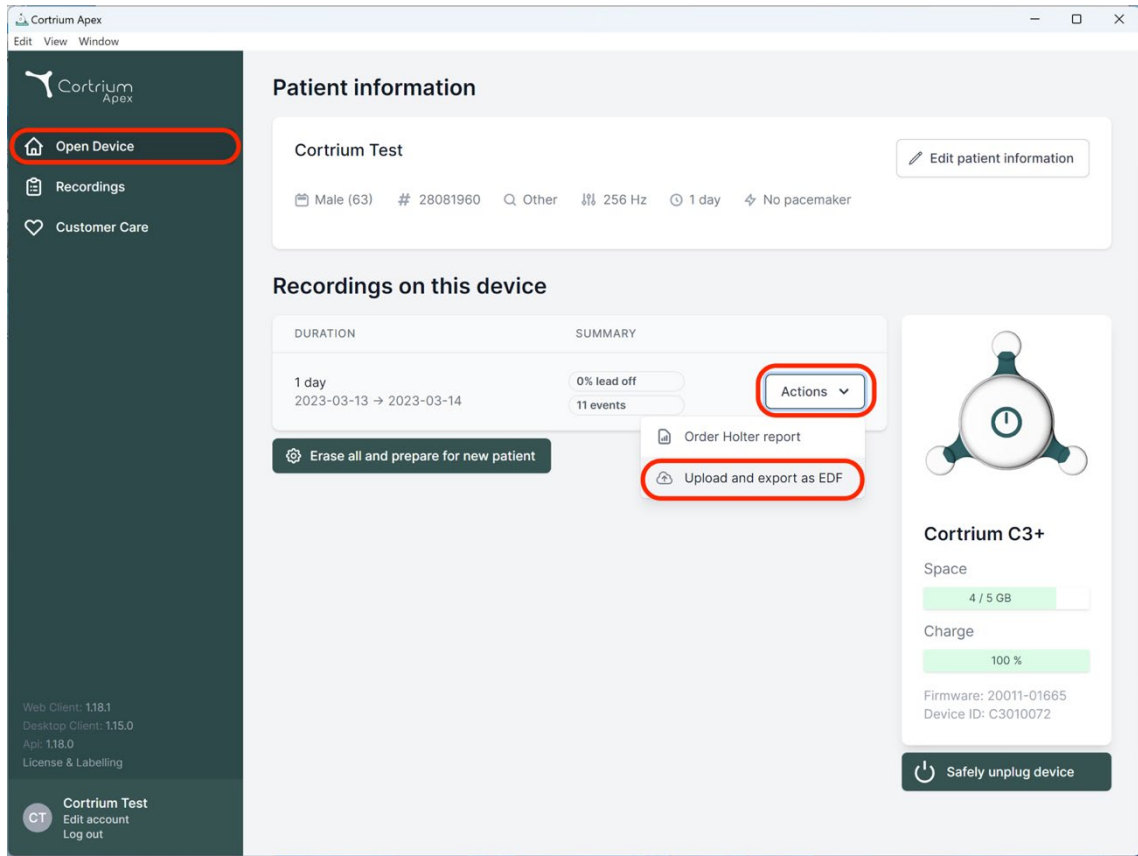
No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

Erase all and prepare for new patient

Cortrium C3+

Space

4 / 5 GB

Charge

100 %

Firmware: 20011-01665

Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Cortrium Apex

Edit View Window

Open Device

Recordings

Customer Care

Web Client: 1.18.1

Desktop Client: 1.15.0

Api: 1.18.0

License & Labelling

Cortrium Test

Edit account

Log out

Patient information

Cortrium Test

Edit patient information

Male (63)

28081960

Q Other

256 Hz

1 day

No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

Erase all and prepare for new patient

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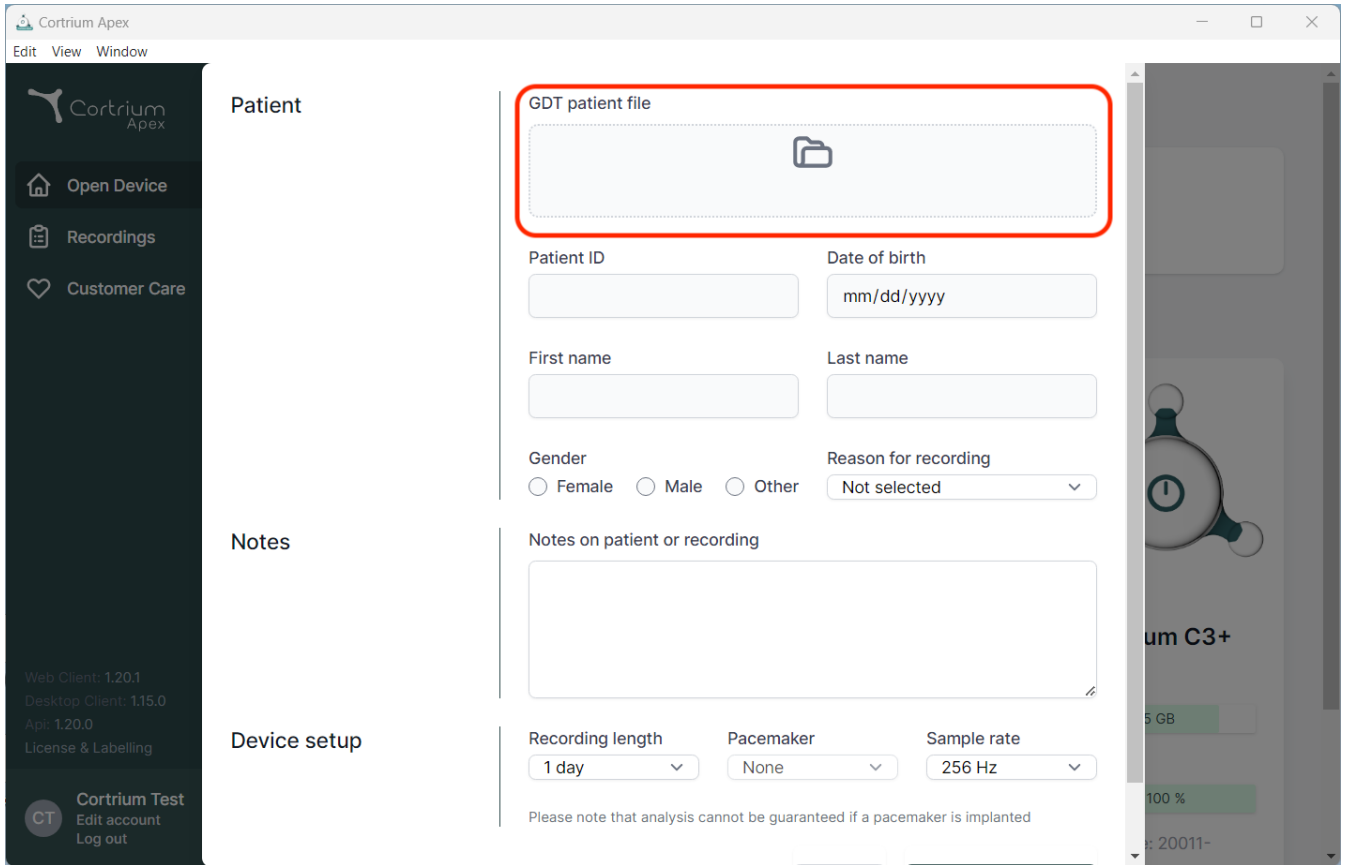
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup.

Patient Section:

- GDT patient file:** A red box highlights a file upload area with a folder icon.
- Patient ID:** A text input field.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for Female, Male, and Other.
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes Section:

- Notes on patient or recording:** A large text area for notes.

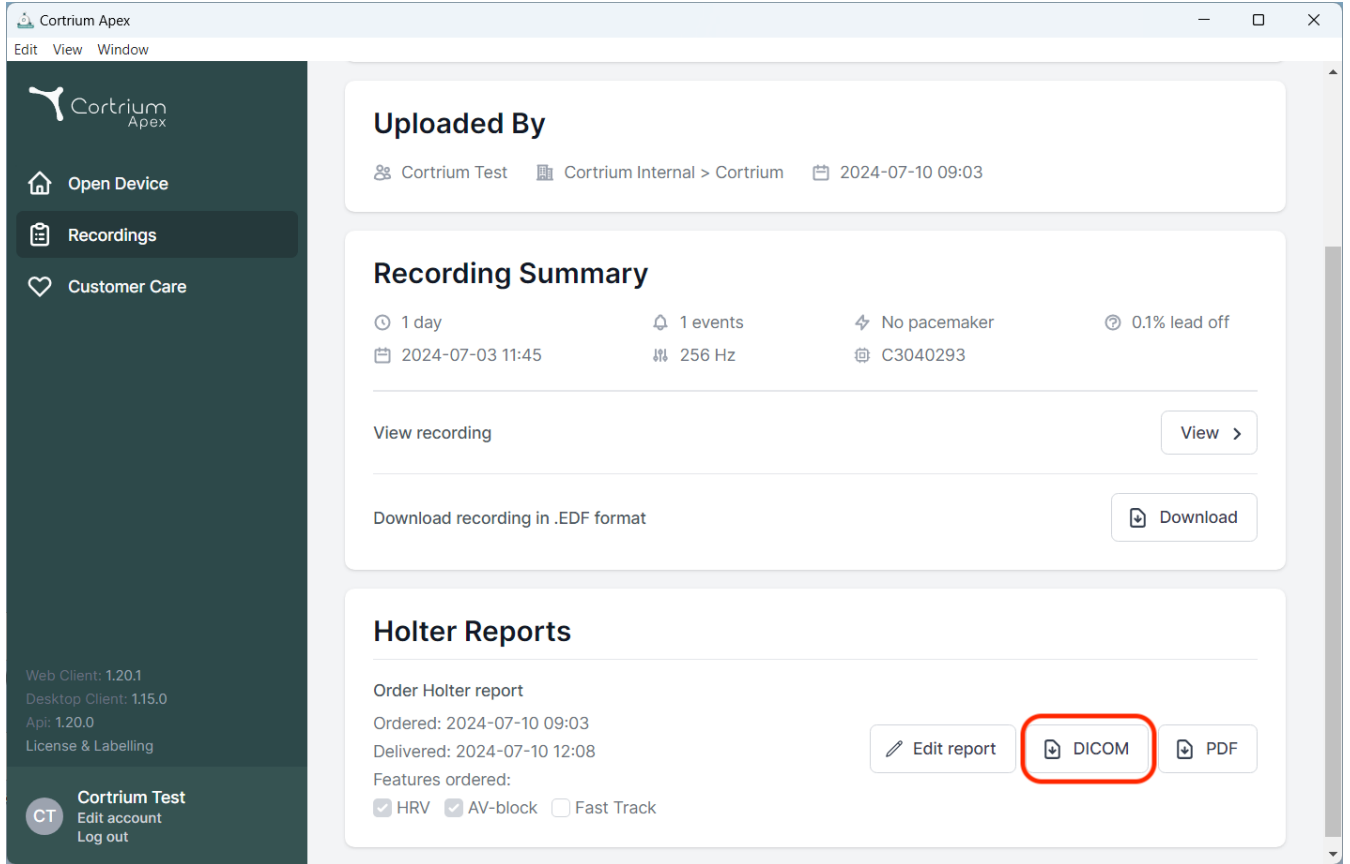
Device setup Section:

- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

At the bottom of the Device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Cortrium Apex' and includes a menu bar with 'Edit', 'View', and 'Window'. The 'Recordings' section is active, showing a recording summary for a device named 'Cortrium Test' on '2024-07-10 09:03'. The summary includes details like '1 day', '1 events', 'No pacemaker', and '0.1% lead off'. Below this, there are buttons for 'View recording' and 'Download recording in .EDF format'. The 'Holter Reports' section is also visible, showing the 'Order Holter report' details, including the order date, delivery date, and features ordered (HRV, AV-block, Fast Track). In this section, the 'DICOM' button is highlighted with a red box, indicating the action to download the DICOM report.

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Account information

Account information

If you change your email address you need to confirm the new address

Title

Initials

CT

First name

Cortrium

Last name

Test

Email address

testreports@cortrium.onmicrosoft.com

Reset password

Account preferences

Here you change preferences for your account

Set preferences

☐

Notify when a Holter report is ready

You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

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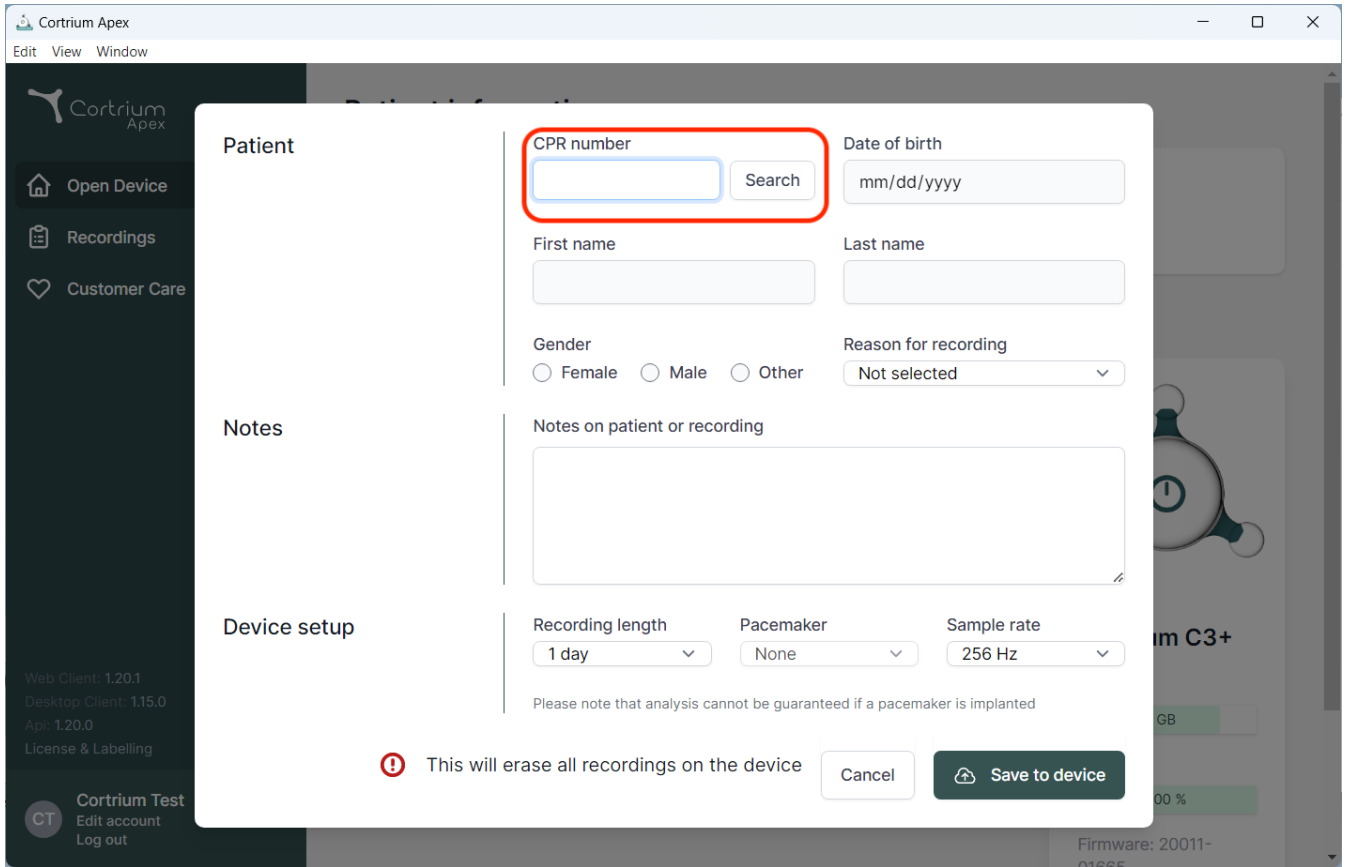
Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Warning: This will erase all recordings on the device

Cancel

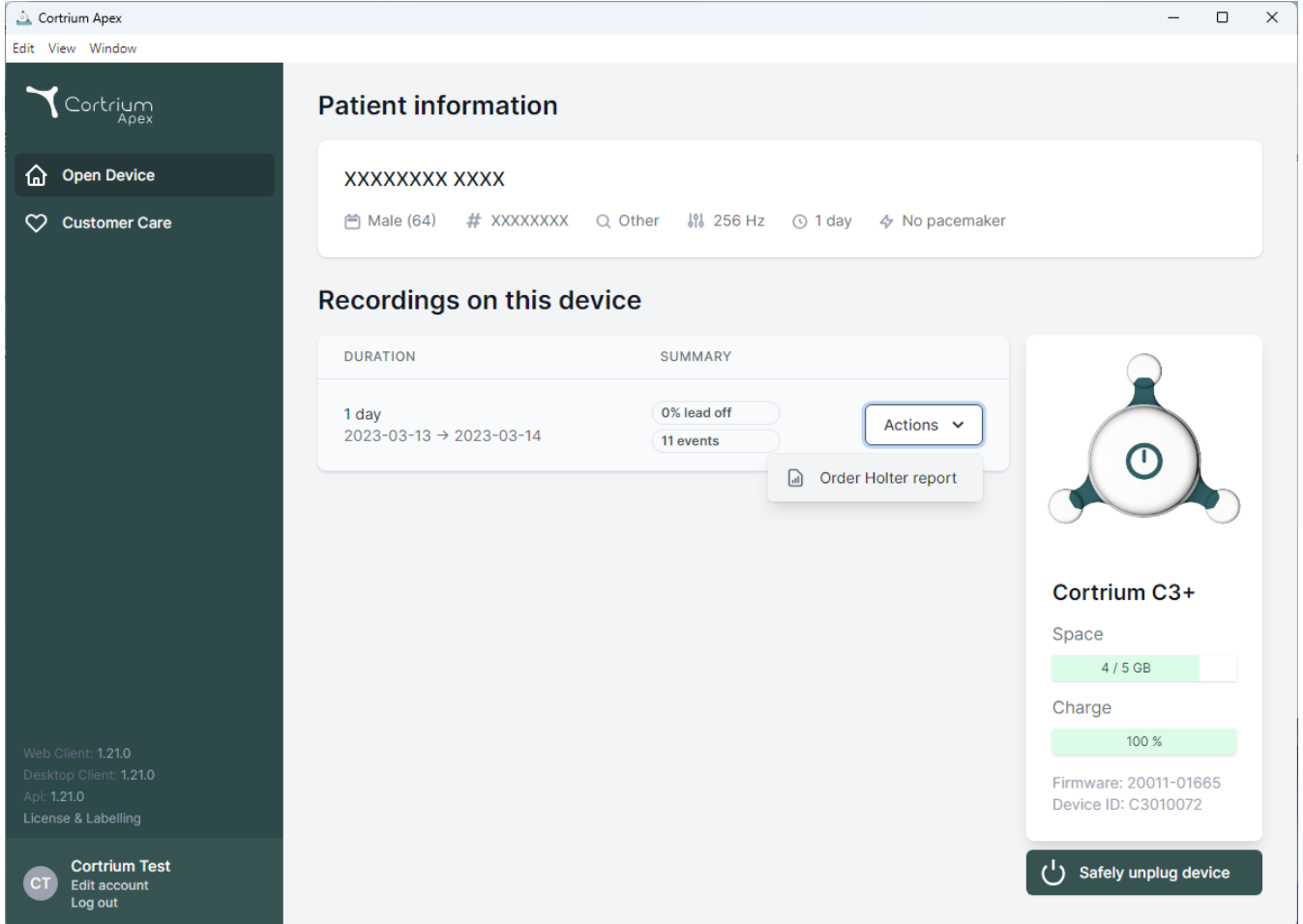
Web Client: 1.20.1
Desktop Client: 1.15.0
App: 1.20.0
License & Labelling

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: 'Open Device' and 'Customer Care'. The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

Patient information

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

Actions

Order Holter report

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

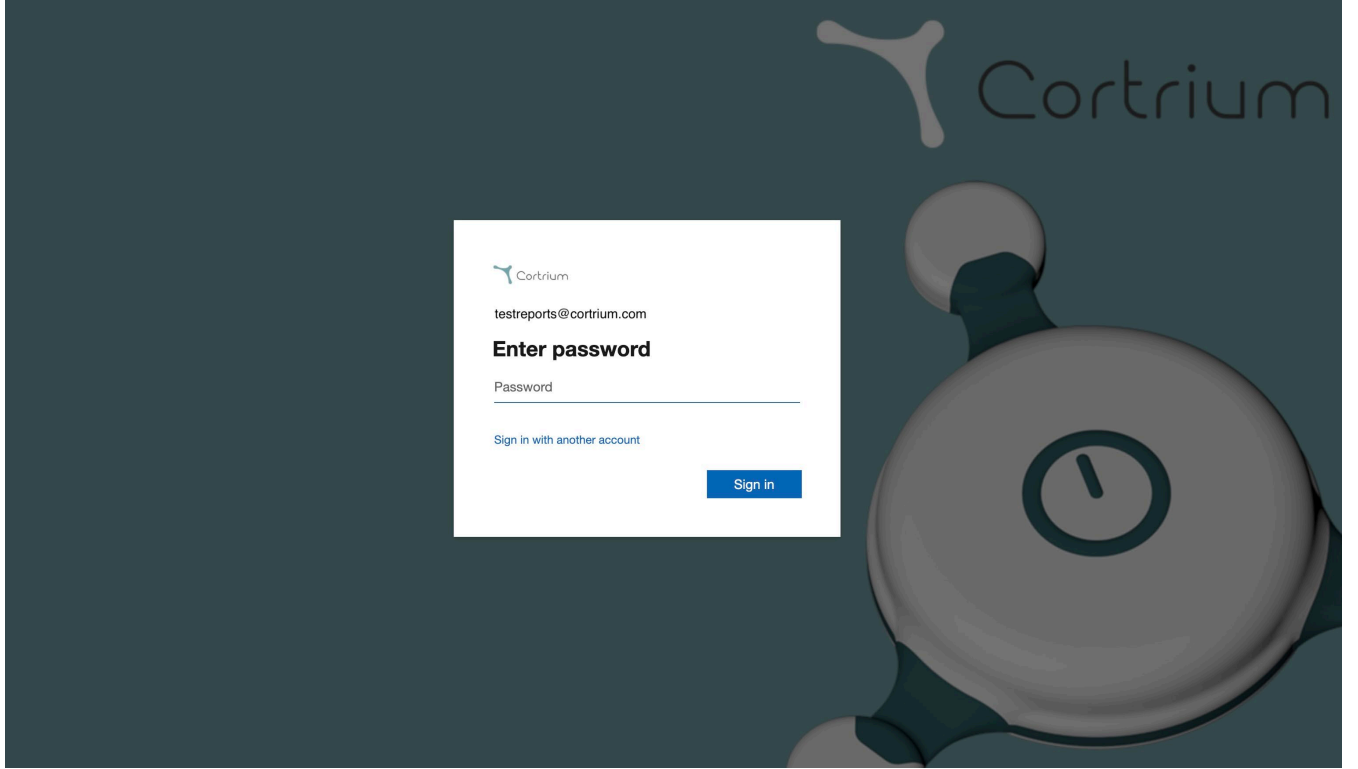
Safely unplug device

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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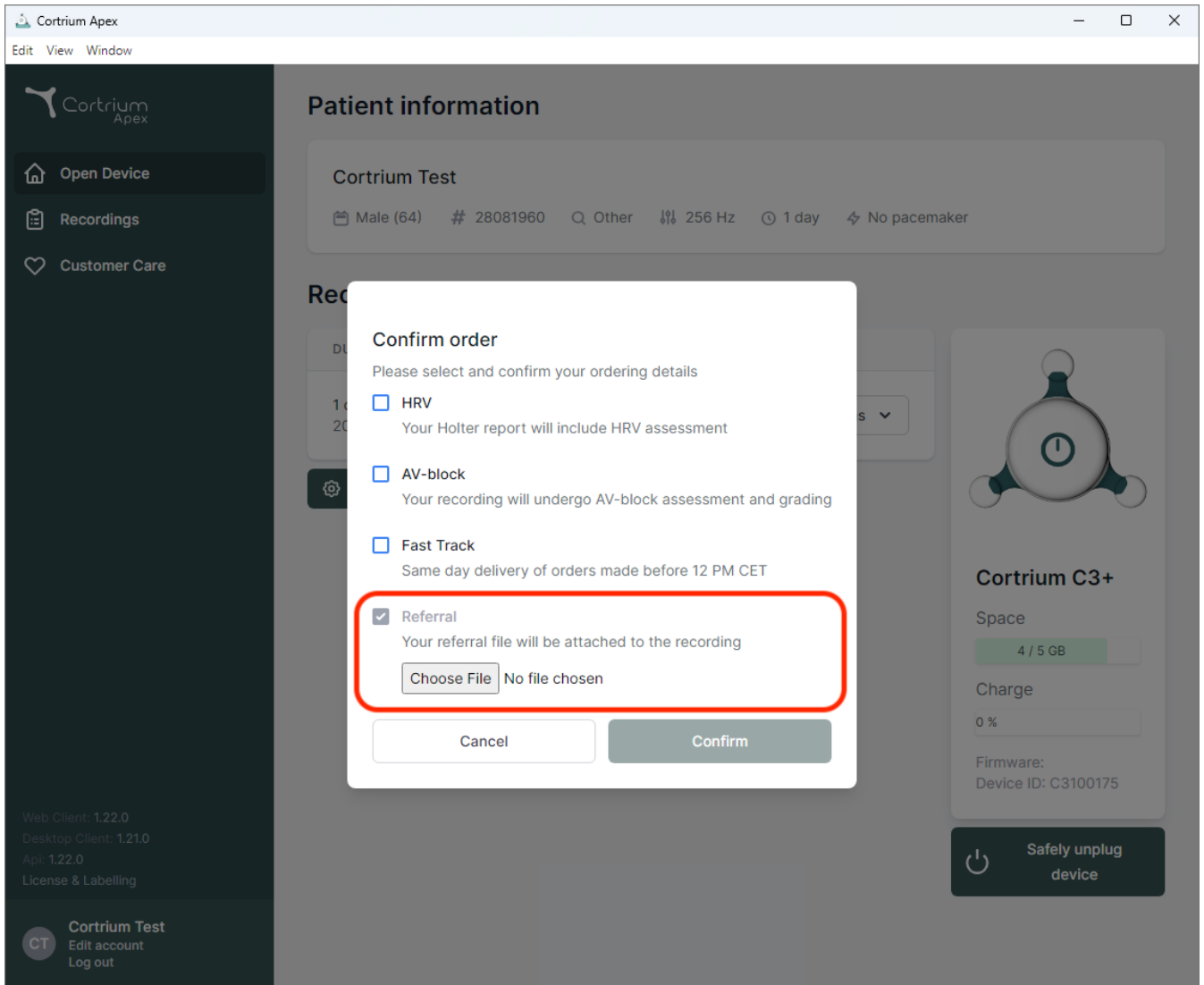
Confirm order
Please select and confirm your ordering details

- ☐ HRV
Your Holter report will include HRV assessment
- ☐ AV-block
Your recording will undergo AV-block assessment and grading
- ☐ Fast Track
Same day delivery of orders made before 12 PM CET
- ☒ Referral
Your referral file will be attached to the recording
[Choose File](#) No file chosen

[Cancel](#) [Confirm](#)

Patient information
Cortrium Test
Male (64) # 28081960 Other 256 Hz 1 day No pacemaker

Cortrium C3+
Space
4 / 5 GB
Charge
0 %
Firmware:
Device ID: C3100175
Safely unplug device

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Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
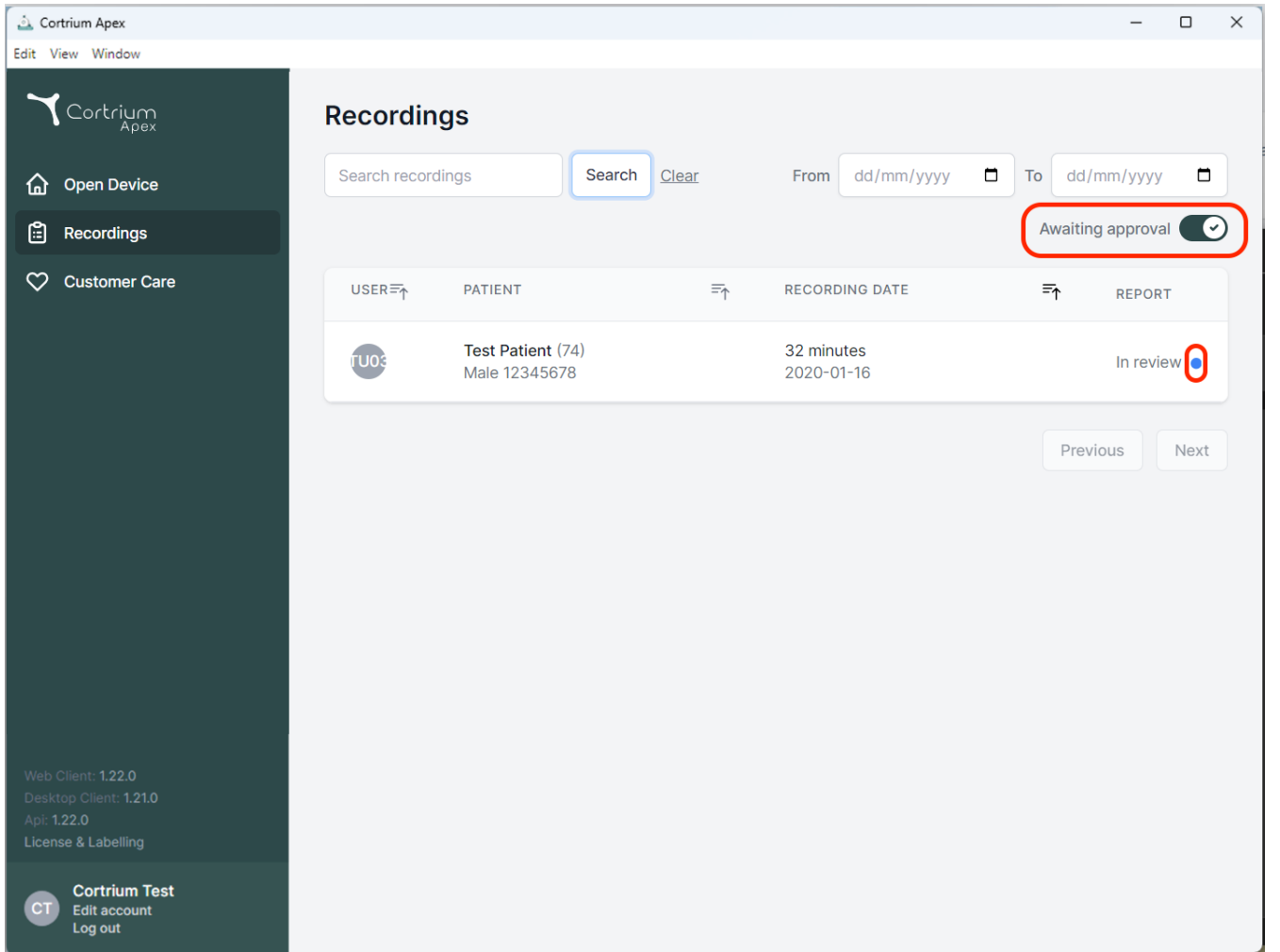
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation options: Open Device, Recordings (selected), and Customer Care. The main content area is titled "Recordings" and includes a search bar, a "Search" button, and a "Clear" link. There are also date pickers for "From" and "To" dates. A toggle switch labeled "Awaiting approval" is highlighted with a red box and is currently turned on. Below this, a table displays recording details:

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

At the bottom right of the table, there are "Previous" and "Next" buttons. The bottom left corner of the interface shows version information: Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, and License & Labelling. The bottom right corner shows the user's name "Cortrium Test" and options to "Edit account" and "Log out".

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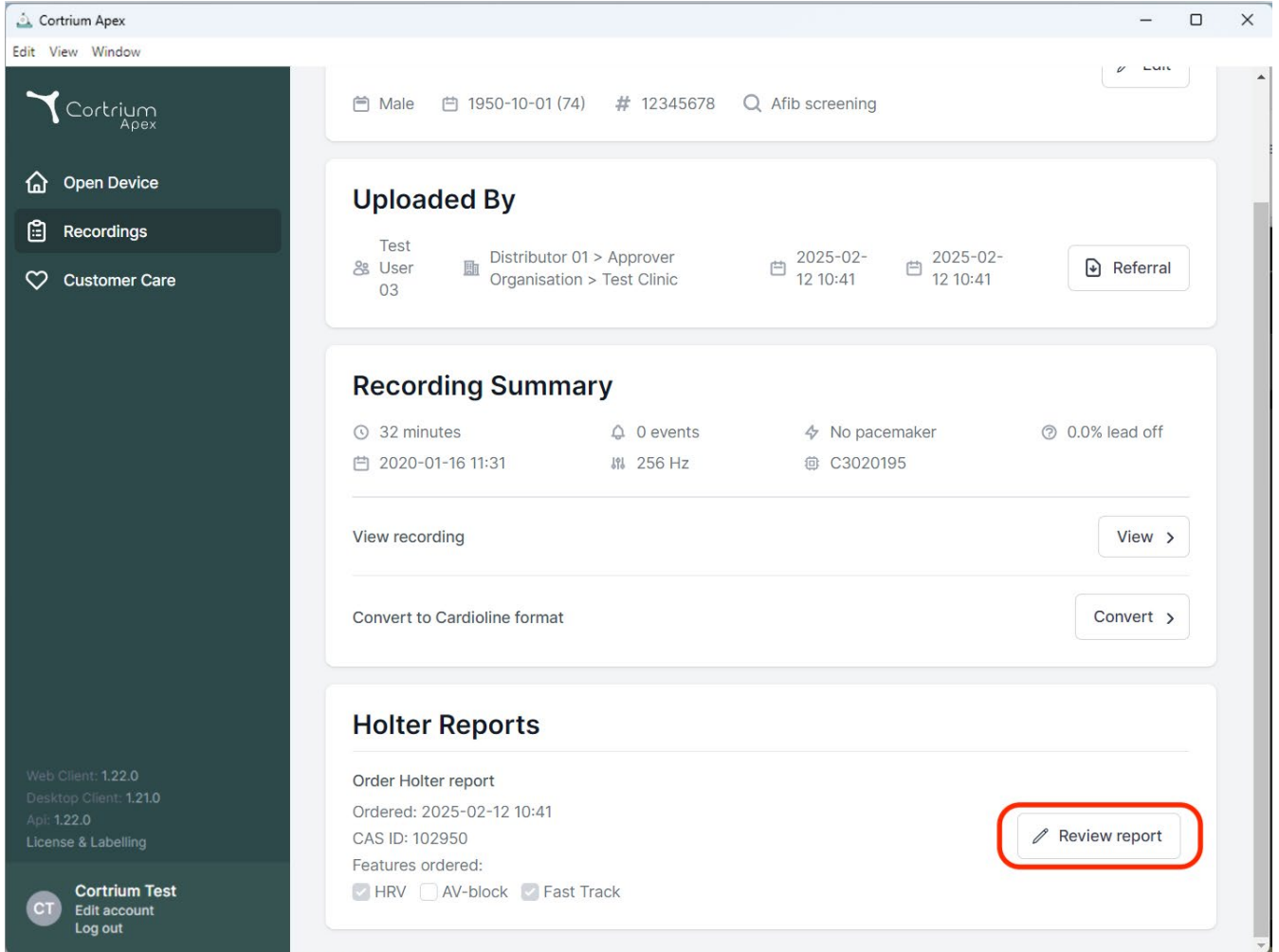The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes the Cortrium Apex logo and a menu with 'Edit', 'View', and 'Window' options. The main content area is divided into several sections:

- Header Information:** Patient details including 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'.
- Uploaded By:** A section showing the test user (User 03), distributor (Distributor 01 > Approver Organisation > Test Clinic), and dates (2025-02-12 10:41). A 'Referral' button is present.
- Recording Summary:** A section providing key recording metrics: 32 minutes, 0 events, No pacemaker, and 0.0% lead off. It also shows the recording date (2020-01-16 11:31) and frequency (256 Hz). Buttons for 'View recording' and 'Convert to Cardioline format' are available.
- Holter Reports:** A section for ordering and reviewing holter reports. It shows the order date (2025-02-12 10:41), CAS ID (102950), and features ordered (HRV, AV-block, Fast Track). A 'Review report' button is highlighted with a red box.

The bottom left sidebar contains the Cortrium Test logo and links for 'Edit account' and 'Log out'. The bottom right corner shows the version information: Web Client: 1.22.0, Desktop Client: 1.21.0, and Api: 1.22.0.

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 Edit View Window

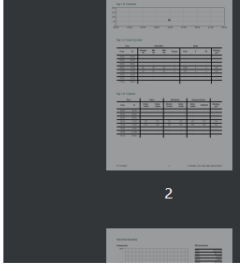
Open Device
 Recordings
 Customer Care

Web Client: 1.22.0
 Desktop Client: 1.21.0
 Api: 1.22.0
 License & Labelling

Cortrium Test
 Edit account
 Log out

 1

 2

Patient ID: 12345678
 Name: Test Patient
 Date of birth: 1995-10-01
 Gender: Male
 Pacemaker: N/A

Clinic: Test Clinic
 Approved by: Test User 02
 Date: 2025-02-12
 Signature:

Examination overview
 Start: 2020-01-16, 11:31:59
 End: 2020-01-16, 12:04:06
 Duration: 32 min

Beats

Category	Percentage
NSR	100.0%
V	0.0%
SV	0.0%

Rhythms

Category	Percentage
NSR	100.0%
VR	0.0%
SVR	0.0%
AFIB	0.0%
Nondiag	0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2021	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Idioventricular	0	AFIB/AFI	0%

Summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

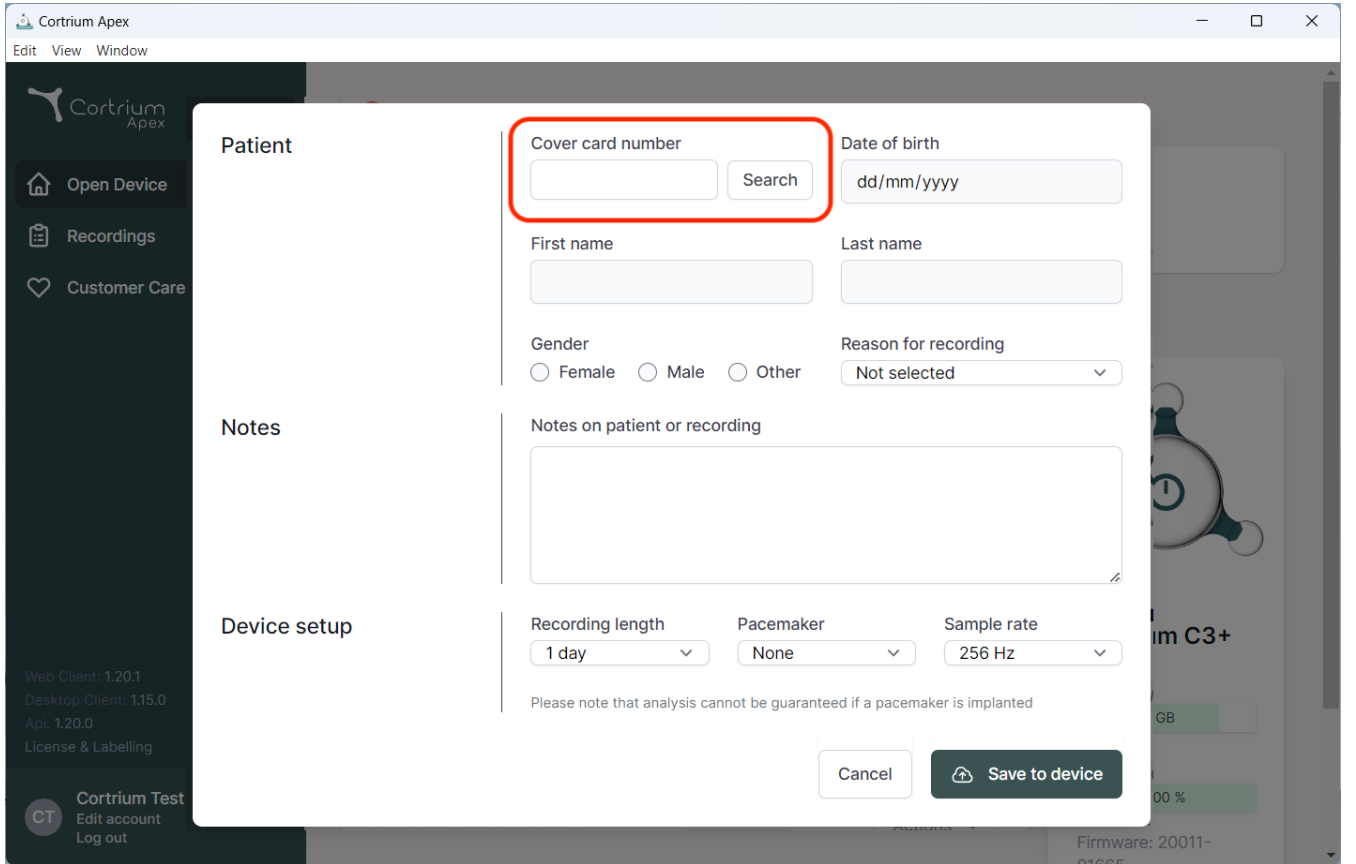
Report has been approved by a cardiologist.
 Name of cardiologist: Test User 02.
 Notes: Some notes.

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Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Cancel

Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling

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