Cortrium Apex 1.27

Cortrium Apex 1.27

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
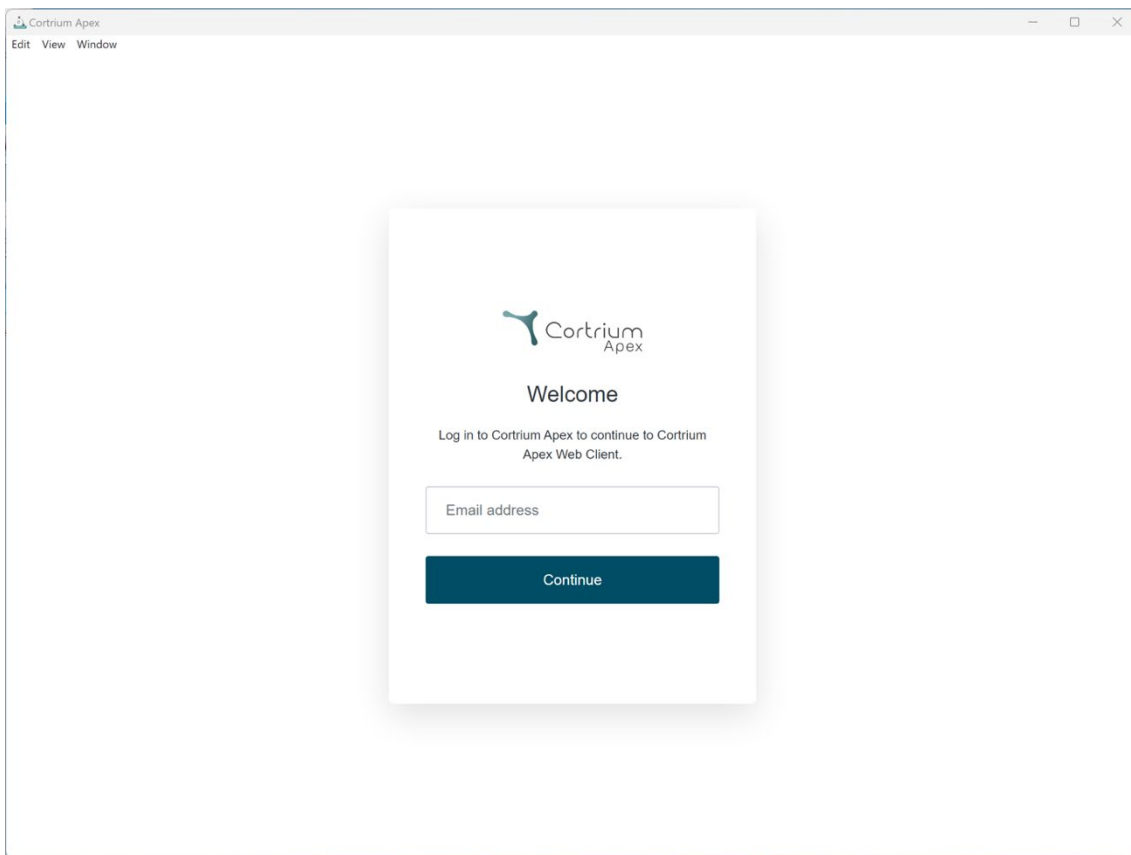
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". The main content area displays a login form centered on the page. The form has a white background and a subtle shadow. At the top of the form is the Cortrium Apex logo. Below the logo is the heading "Welcome". Underneath the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." followed by a text input field labeled "Email address". At the bottom of the form is a dark blue button with the text "Continue" in white.

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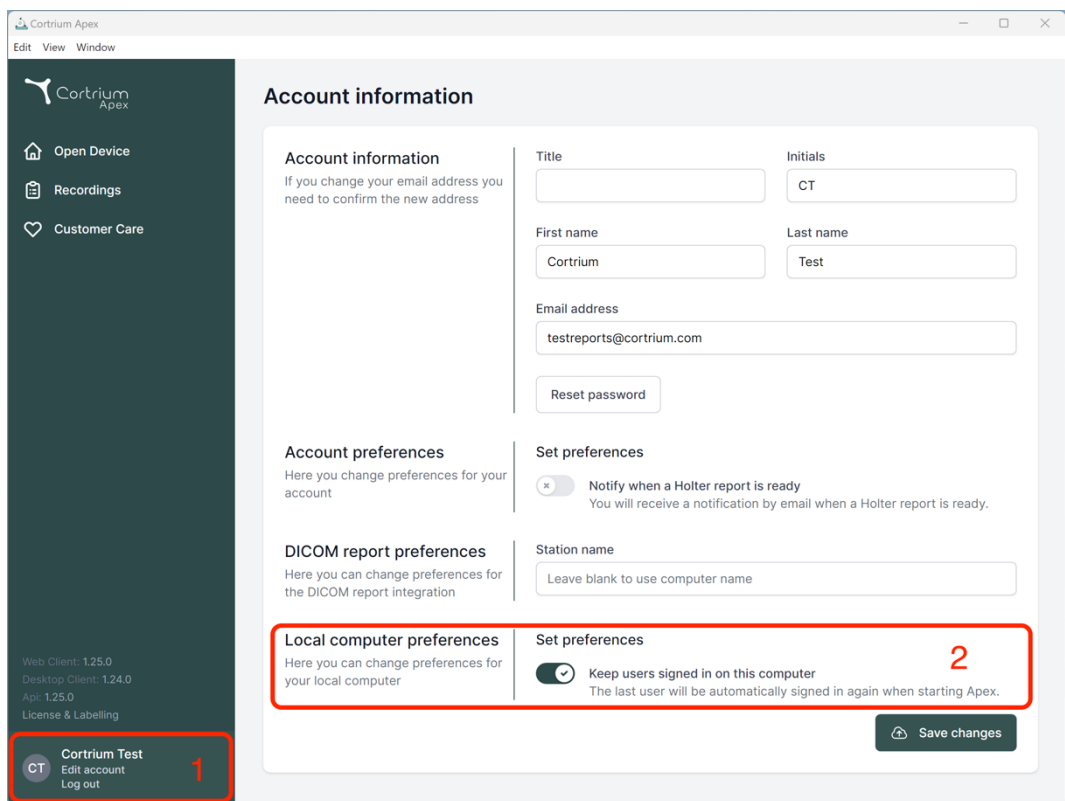
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details: 'Web Client: 1.25.0', 'Desktop Client: 1.24.0', 'Api: 1.25.0', and 'License & Labelling'. At the bottom of the sidebar is a user profile card for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area is titled 'Account information' and contains several sections: 'Account information' (with fields for Title, Initials, First name, Last name, and Email address), 'Account preferences' (with a toggle for 'Notify when a Holter report is ready'), 'DICOM report preferences' (with a field for 'Station name'), and 'Local computer preferences' (with a toggle for 'Keep users signed in on this computer'). A red box highlights the 'Local computer preferences' section, and another red box highlights the 'Set preferences' section for local computer preferences. A 'Save changes' button is located at the bottom right of the main content area.

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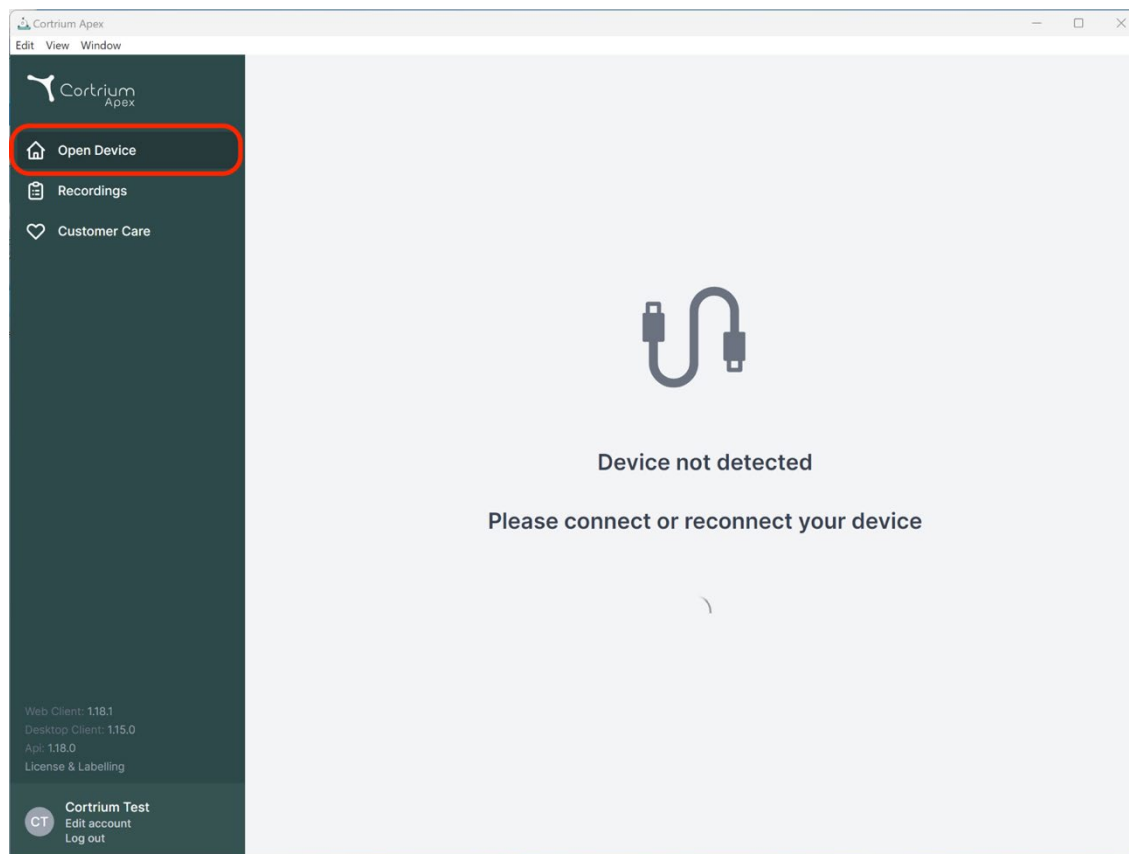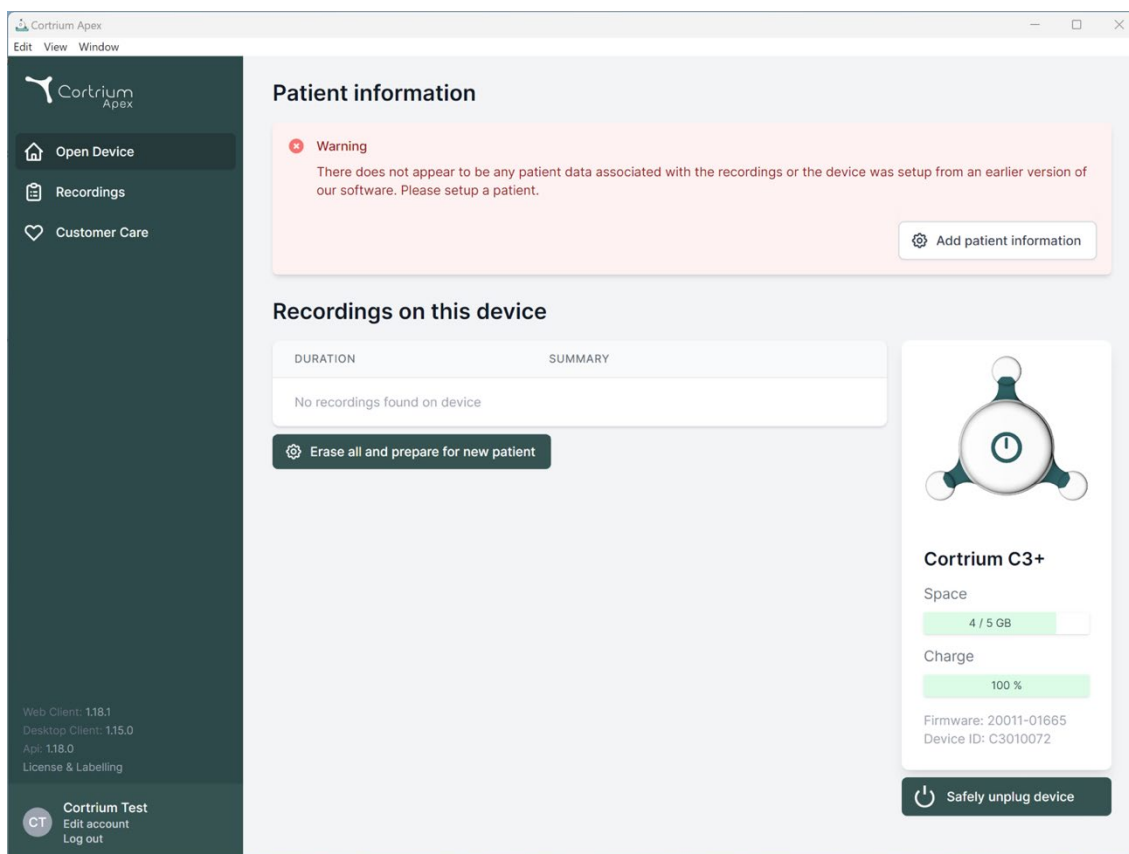
Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Save to device

Warning This will erase all recordings on the device

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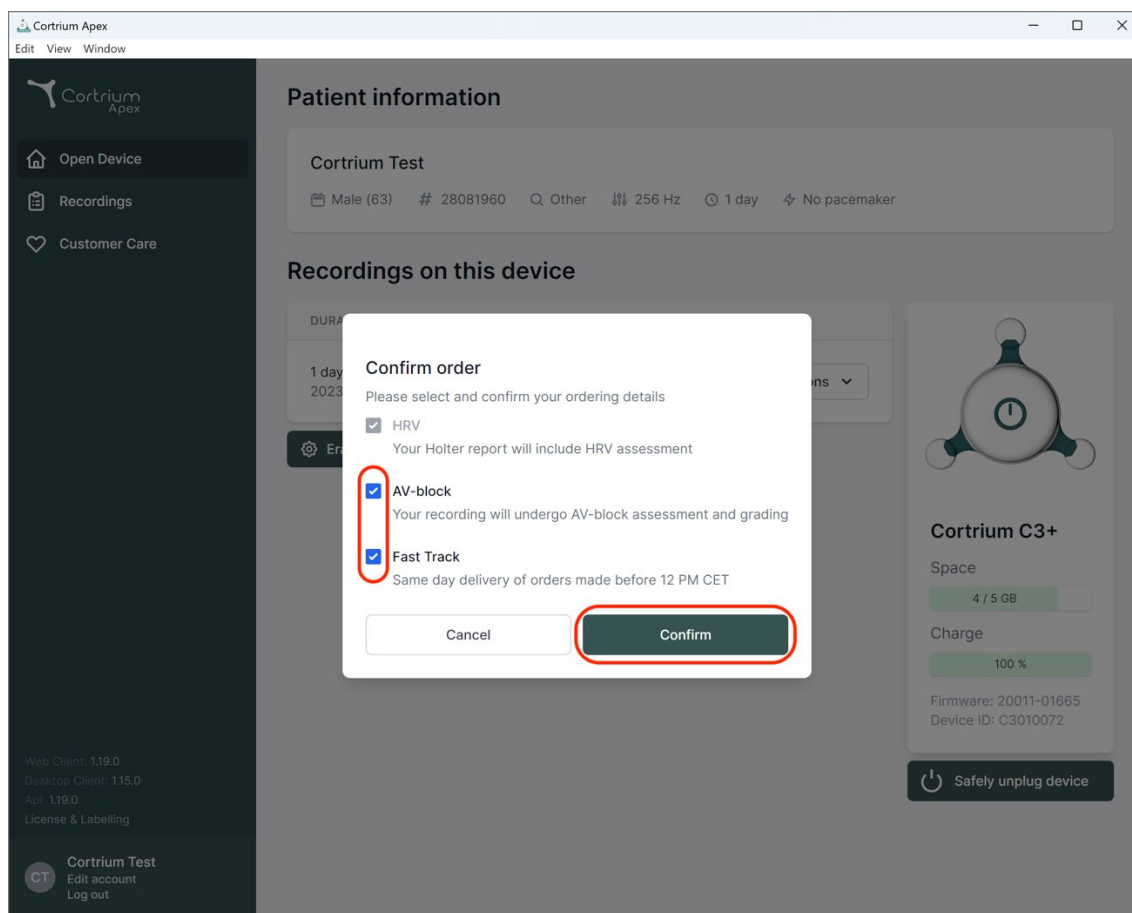
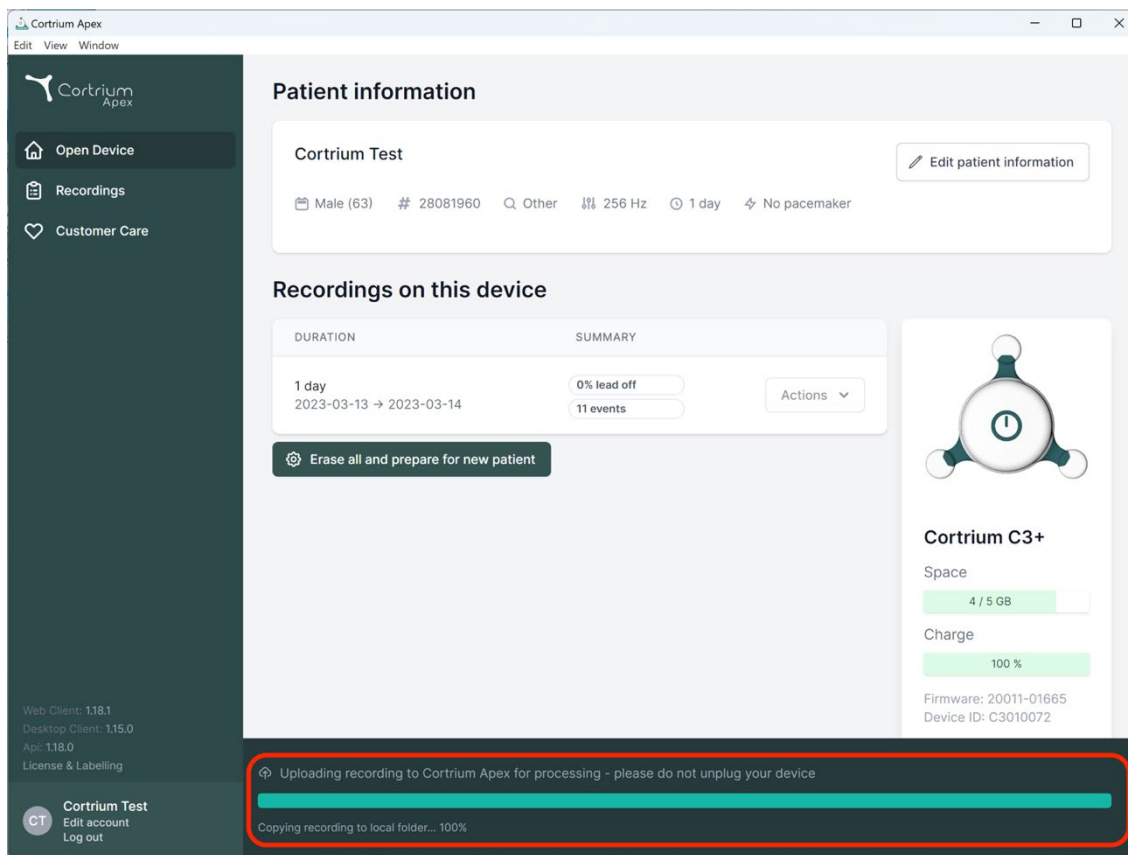
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and recording duration (1 day). Below this, the 'Recordings on this device' section shows a table with columns for DURATION and SUMMARY. The current recording is for 1 day (2023-03-13 to 2023-03-14) with 0% lead off and 11 events. A button 'Erase all and prepare for new patient' is visible. On the right, the 'Cortrium C3+' device status is shown, including space (4 / 5 GB) and charge (100%). At the bottom, a red-bordered box highlights a status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and a progress bar for 'Copying recording to local folder... 100%'. The bottom left of the sidebar shows the user's role as 'Cortrium Test' with options to edit the account or log out.

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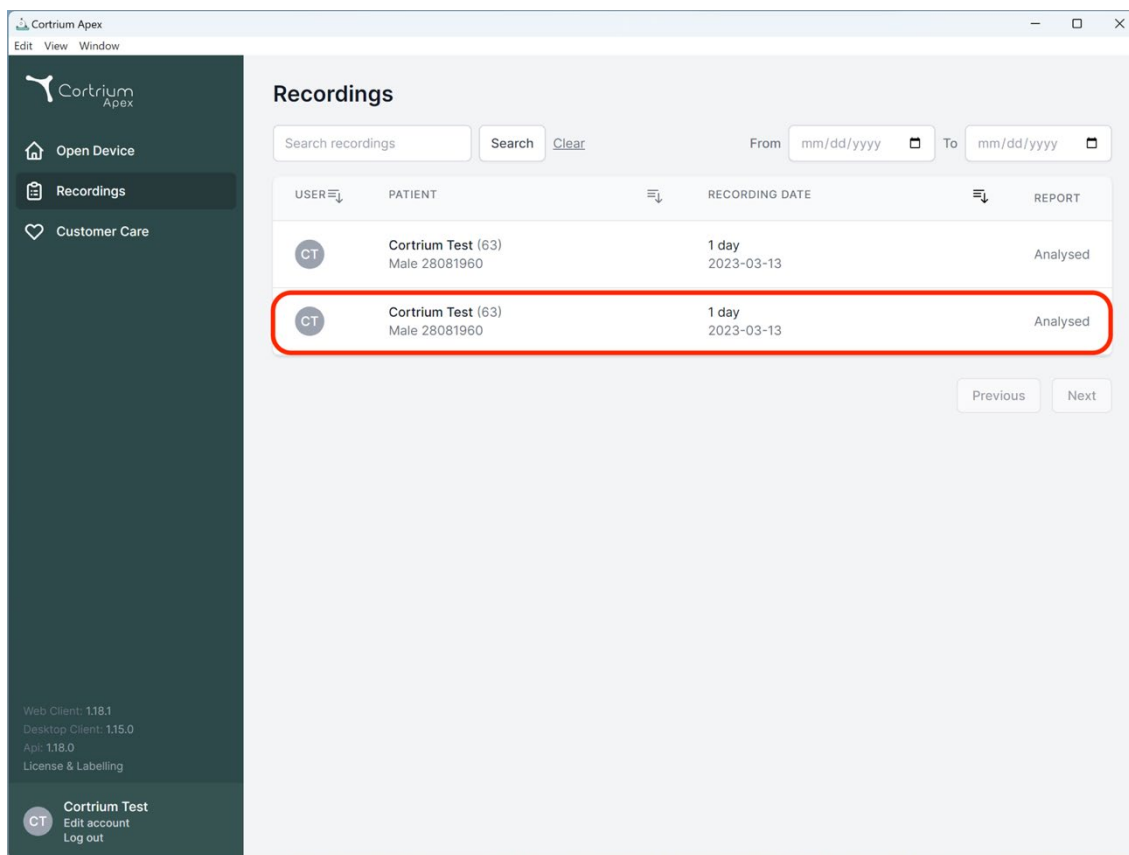
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (active), and 'Customer Care'. At the bottom of the sidebar, it shows 'Web Client: 118.1', 'Desktop Client: 115.0', 'App: 118.0', 'License & Labelling', and a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area has a title 'Recordings' and a search bar. Below the search bar are date filters 'From' and 'To' in 'mm/dd/yyyy' format. A table lists recordings with columns: 'USER' (with a dropdown arrow), 'PATIENT' (with a dropdown arrow), 'RECORDING DATE' (with a dropdown arrow), and 'REPORT'. Two rows are visible, both for 'Cortrium Test (63)' and 'Male 28081960' on '2023-03-13' with a '1 day' duration and 'Analysed' status. The second row is highlighted with a red border. At the bottom right of the table are 'Previous' and 'Next' buttons.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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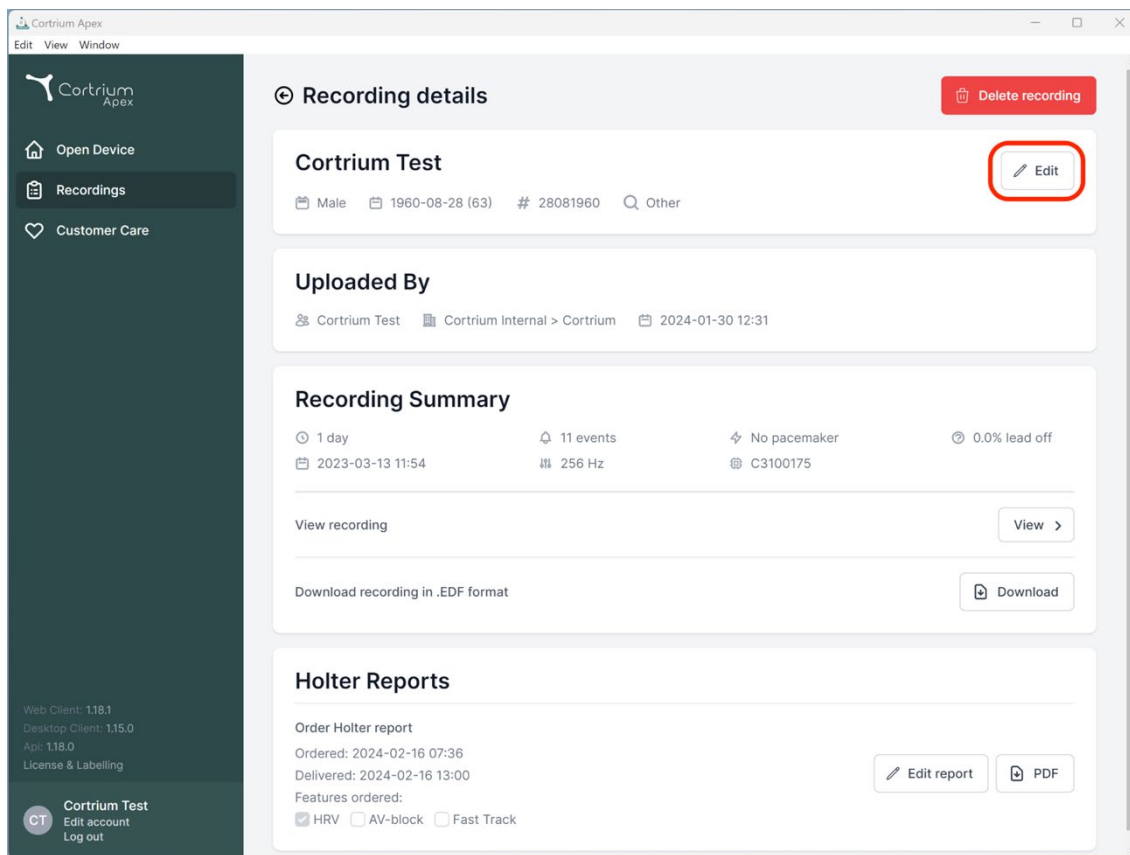
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

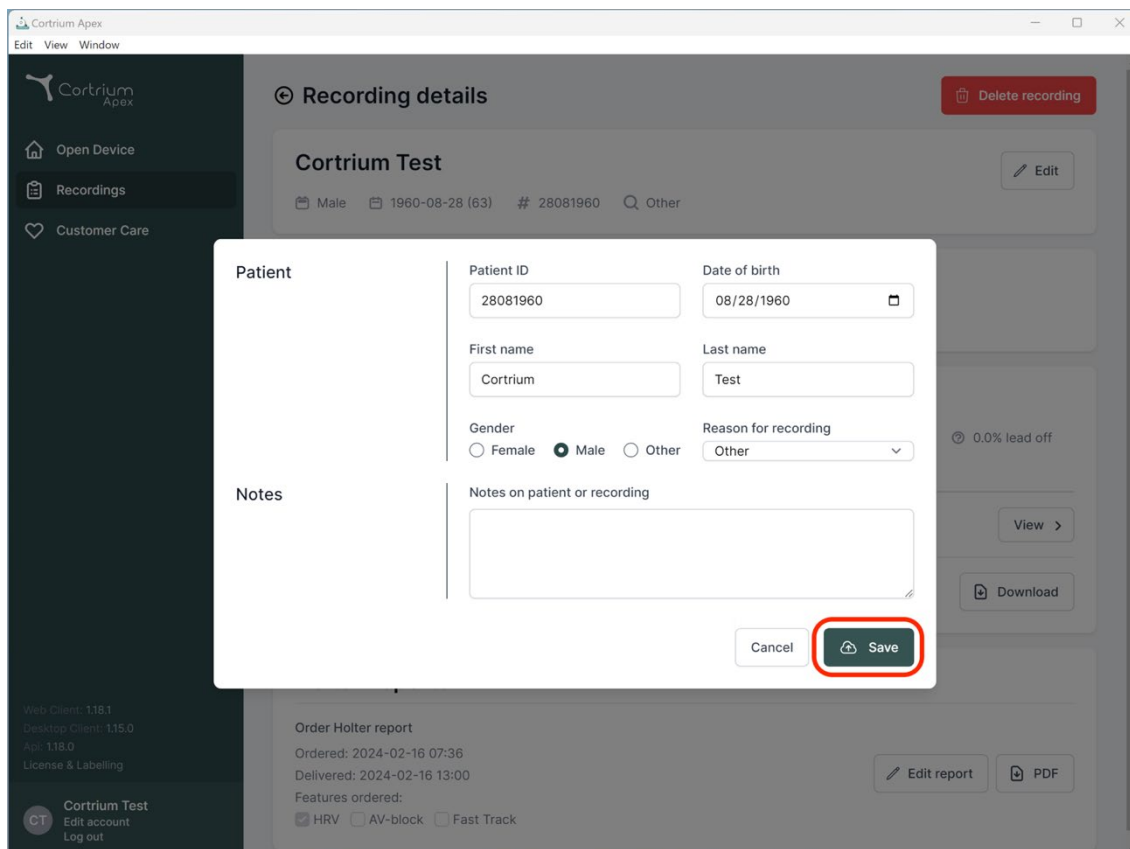
Order Holter report

Ordered: 2024-02-16 07:36
 Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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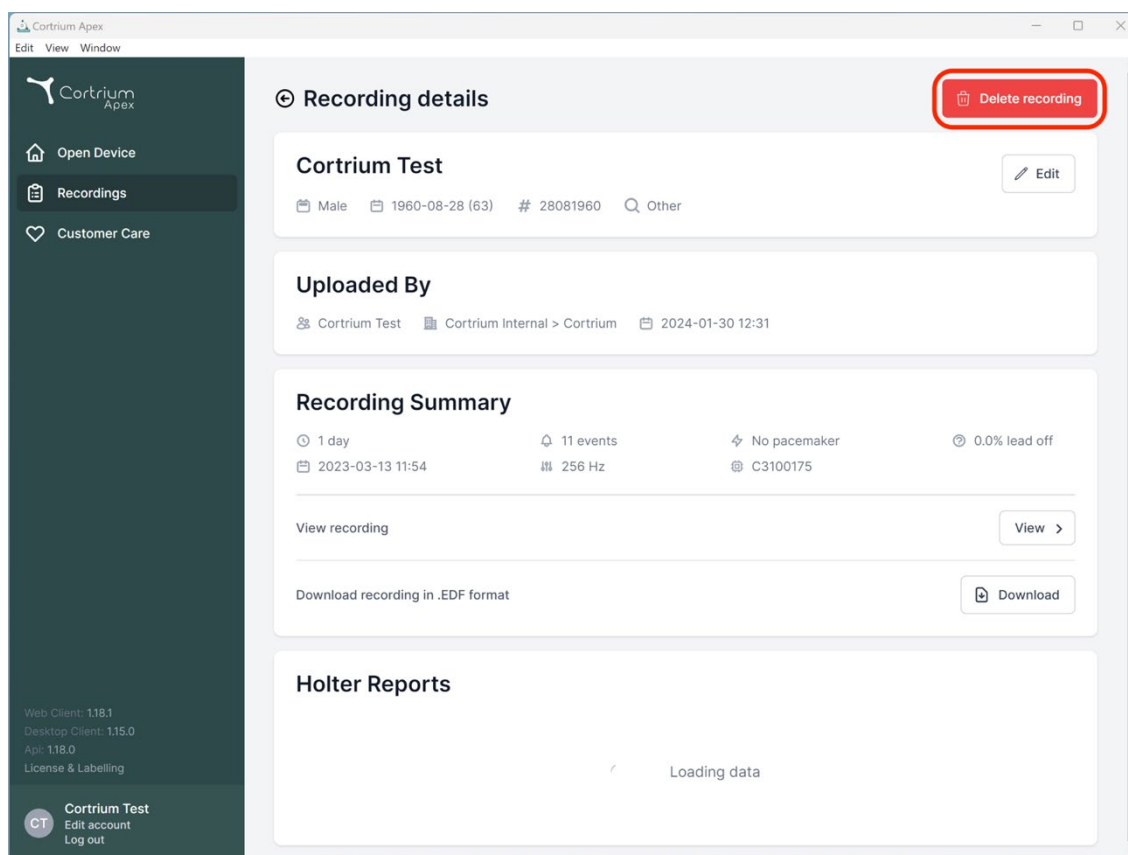
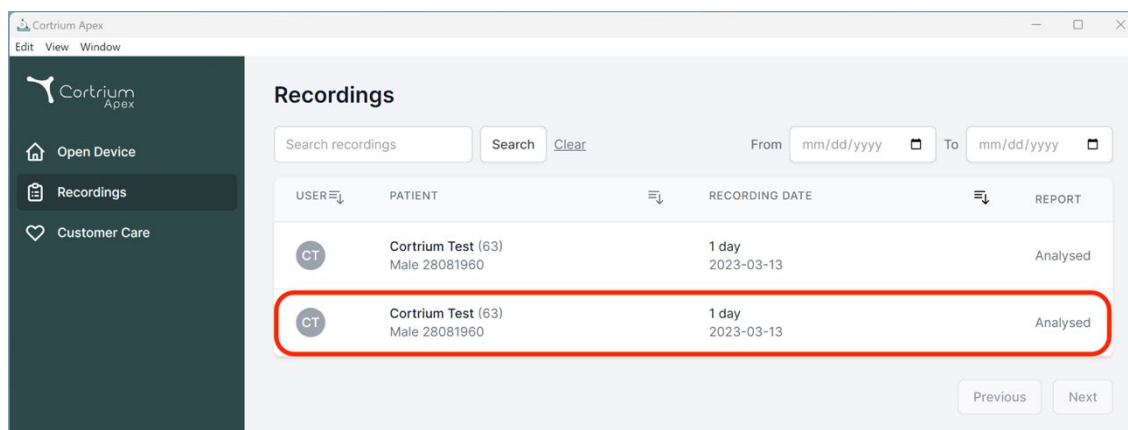
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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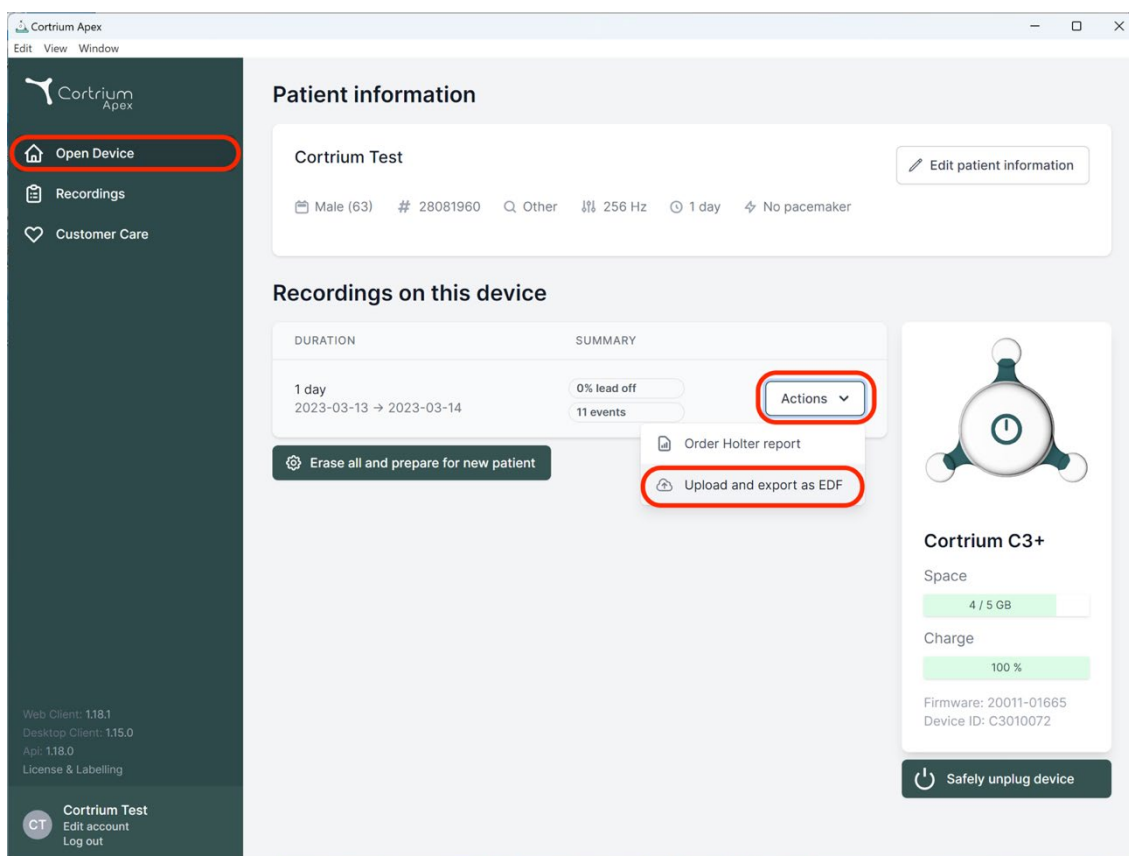
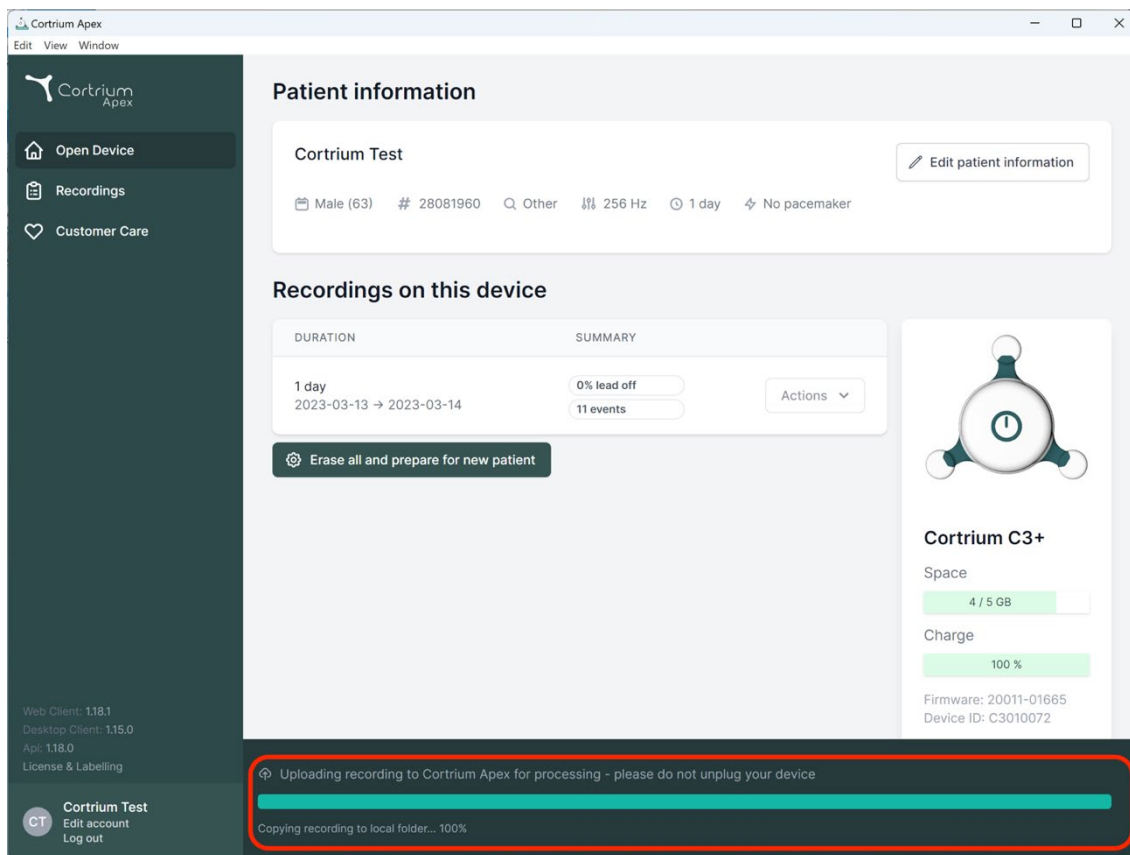
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

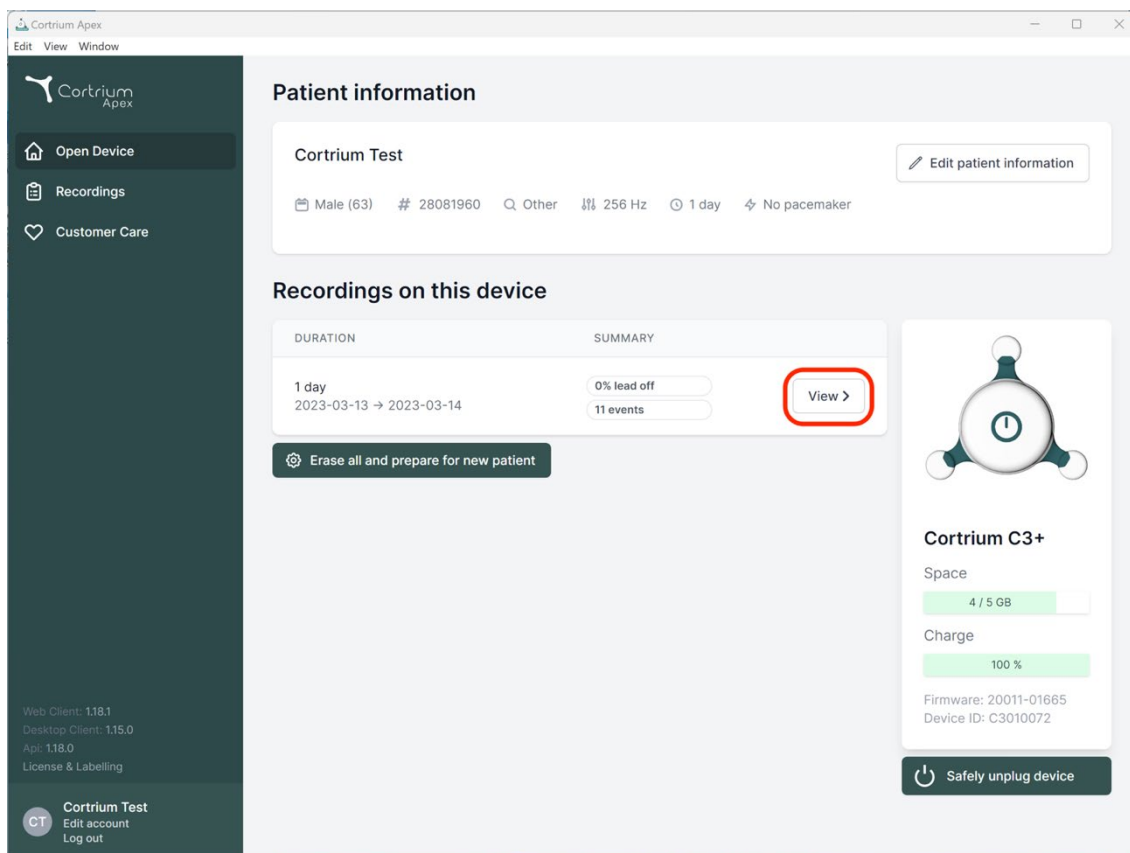
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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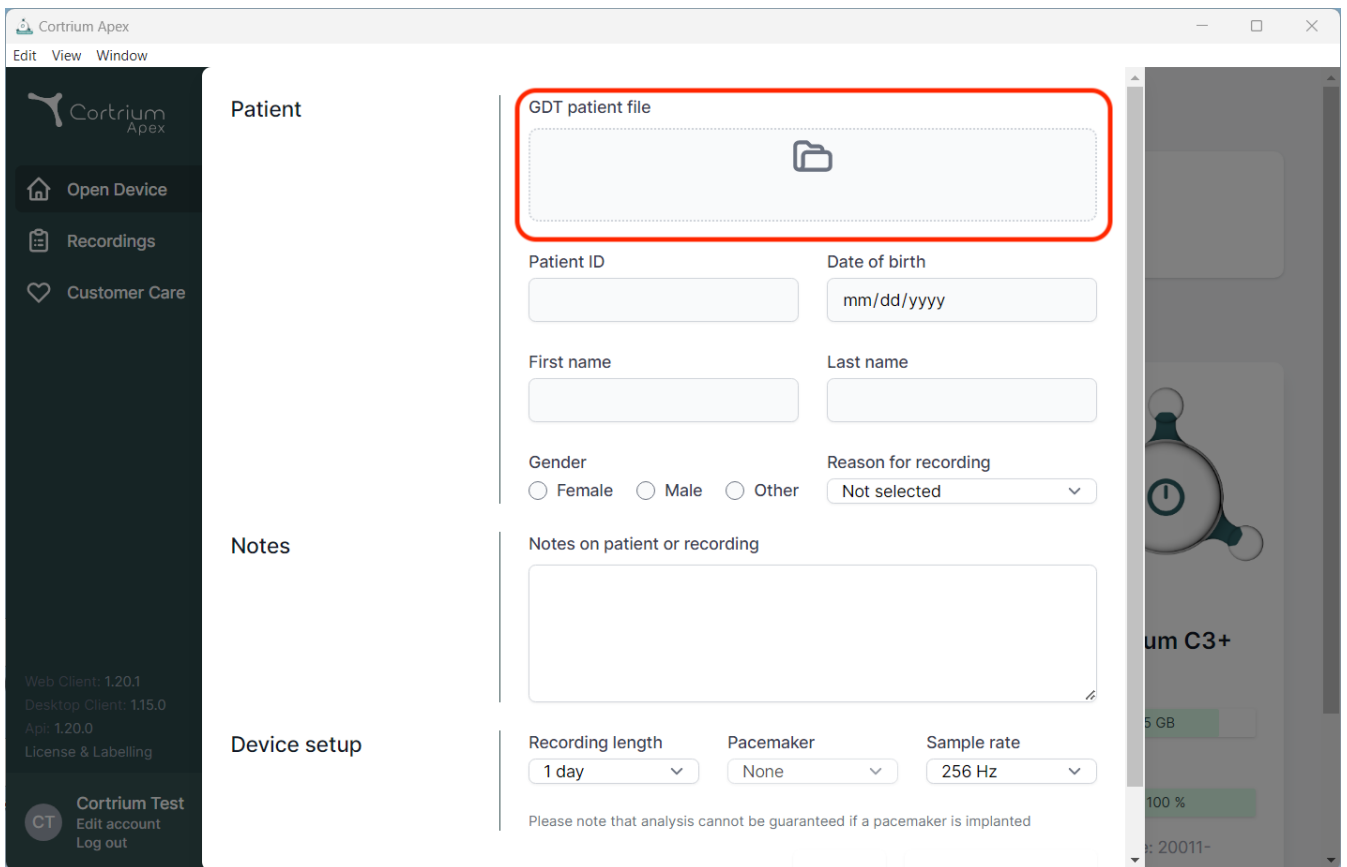
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". The main content area is divided into three sections: "Patient", "Notes", and "Device setup".

Patient section:

- GDT patient file:** A red-bordered box containing a folder icon, indicating where to upload the patient file.
- Patient ID:** A text input field.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes section:

- Notes on patient or recording:** A large text area for entering notes.

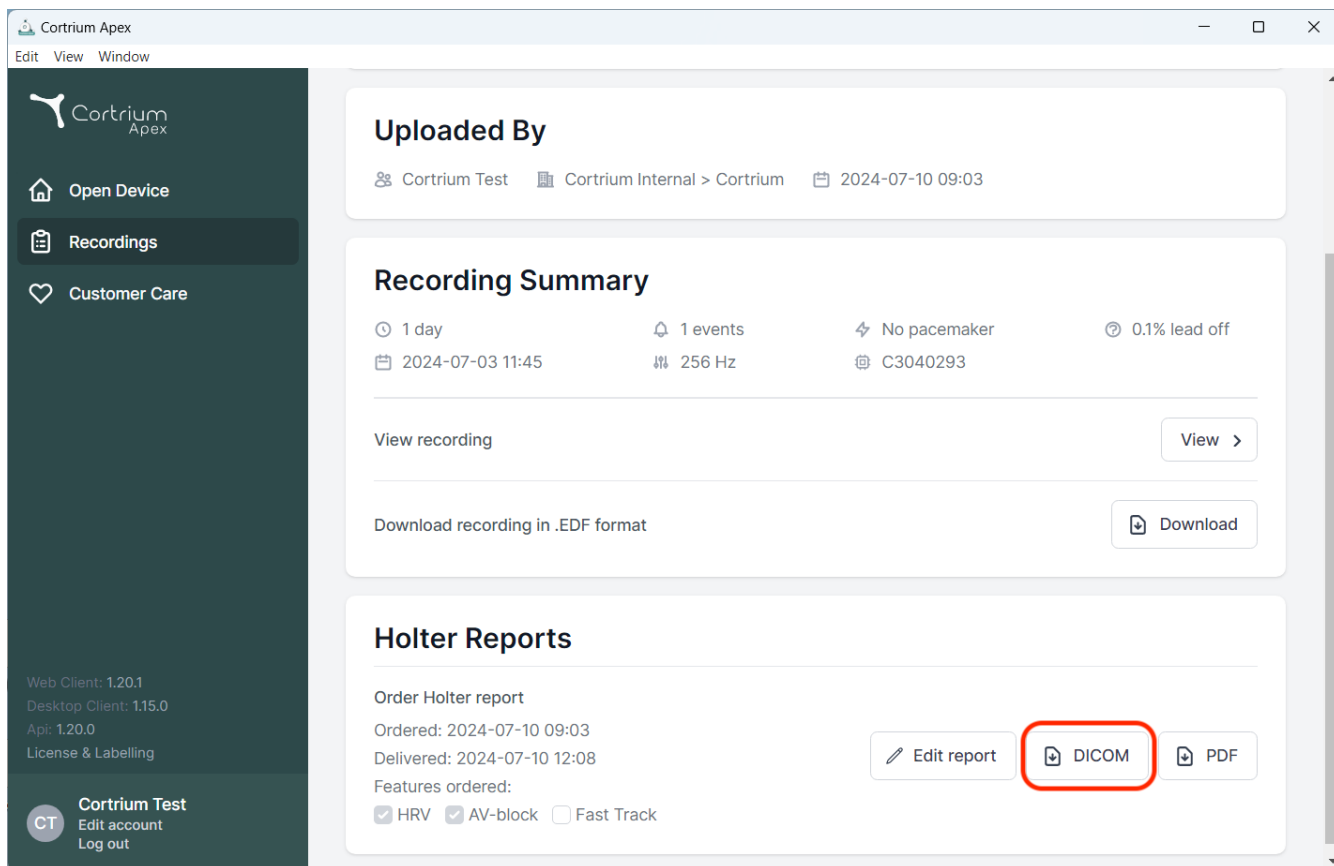
Device setup section:

- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

At the bottom of the "Device setup" section, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' as the uploader, 'Cortrium Internal > Cortrium' as the location, and the date '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' sampling rate, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (for .EDF format).
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red box), and 'PDF'.

At the bottom left of the sidebar, version information is listed: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'Api: 1.20.0', and 'License & Labelling'. A user profile for 'Cortrium Test' is also visible.

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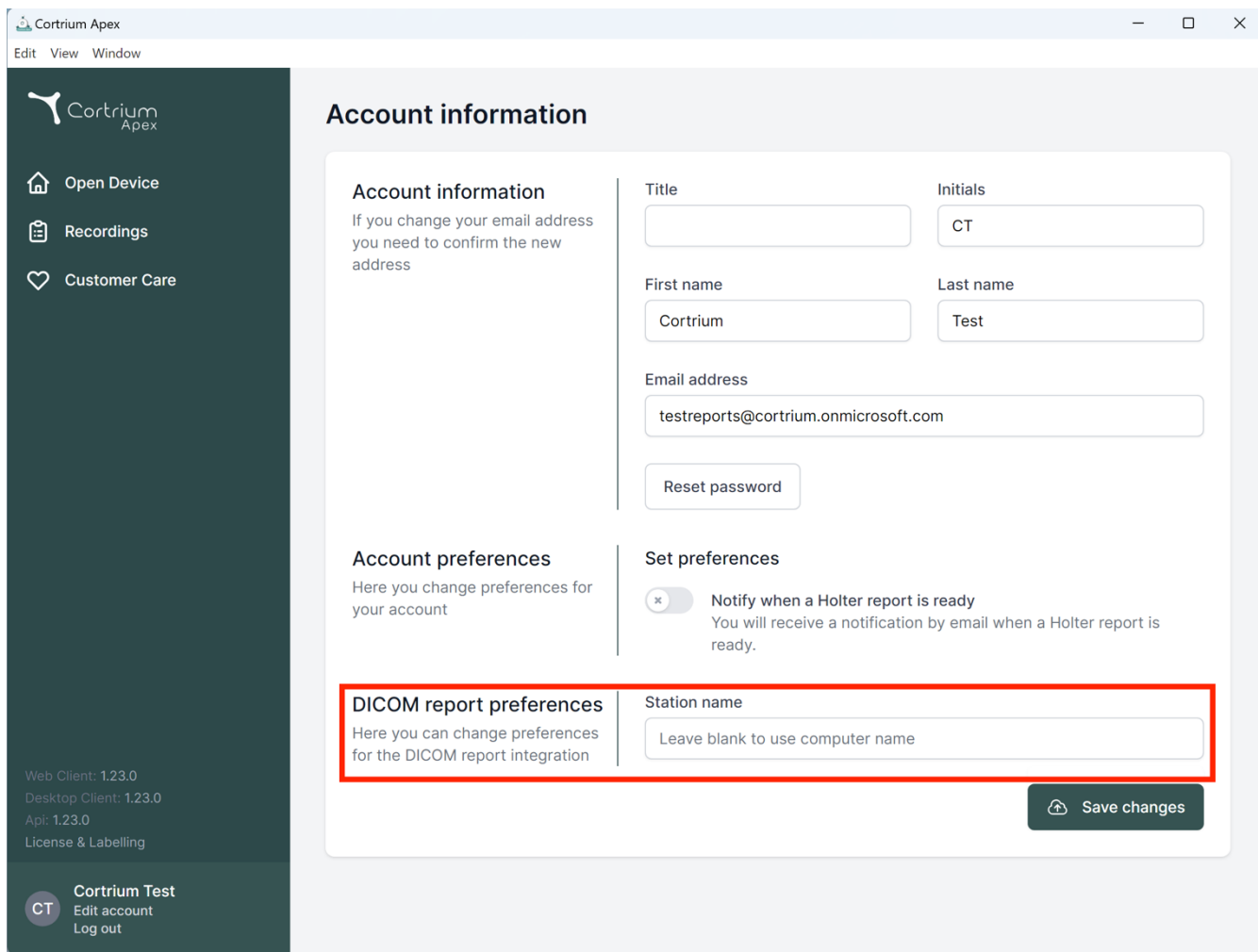Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

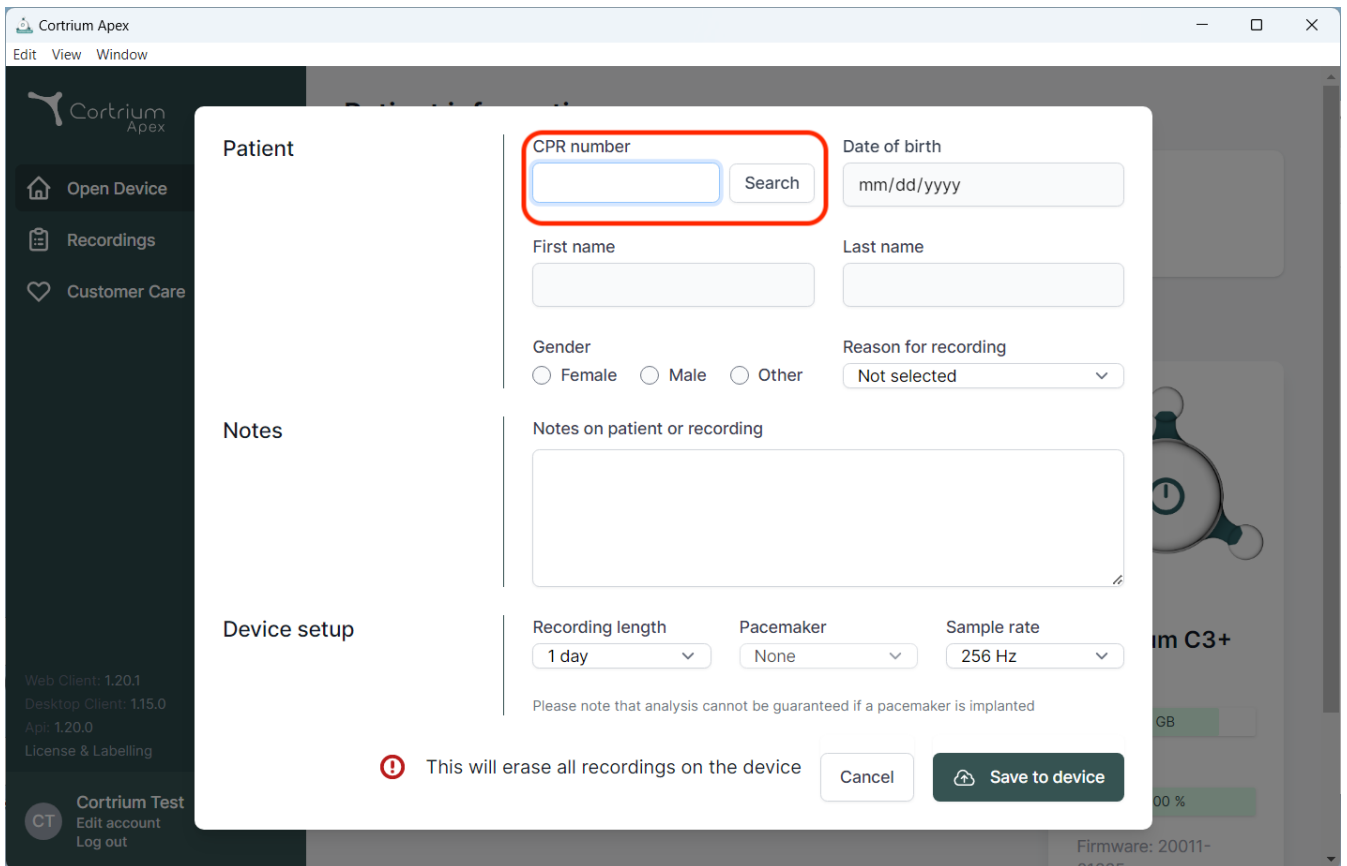
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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
The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient search and recording setup. The "CPR number" field is highlighted with a red box, and a "Search" button is next to it. The form includes fields for "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), "Notes on patient or recording", "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A warning message at the bottom states: "This will erase all recordings on the device". The background shows the Cortrium Apex sidebar with options like "Open Device", "Recordings", and "Customer Care".

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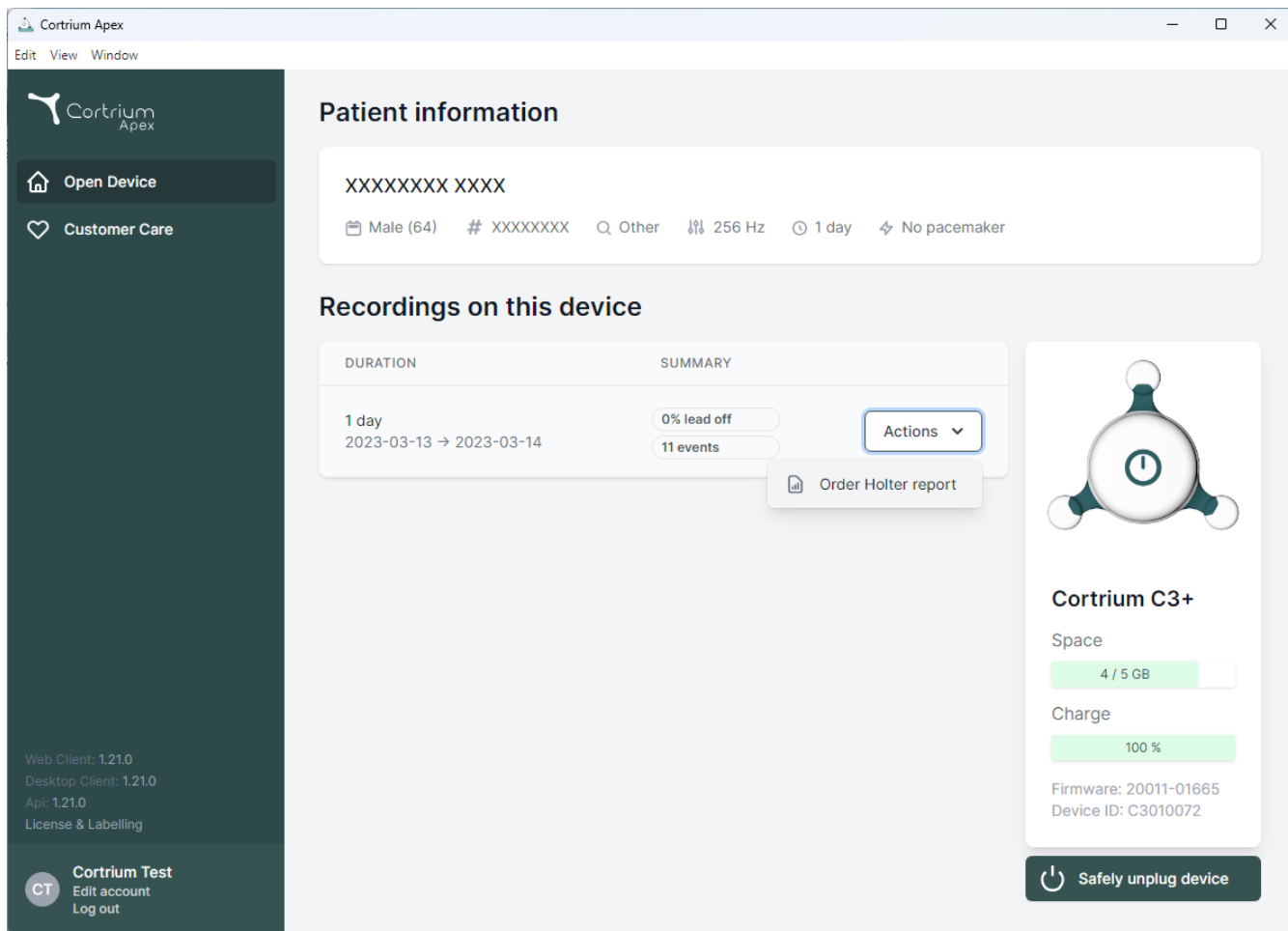
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient profile for 'XXXXXXXX XXXX'. Below the name, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with two columns: 'DURATION' and 'SUMMARY'. The table contains one entry for '1 day' (2023-03-13 → 2023-03-14) with a summary of '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the recording, with an option to 'Order Holter report'. On the right side, a device status card for 'Cortrium C3+' shows 'Space' (4 / 5 GB) and 'Charge' (100 %) levels, along with 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of this card.

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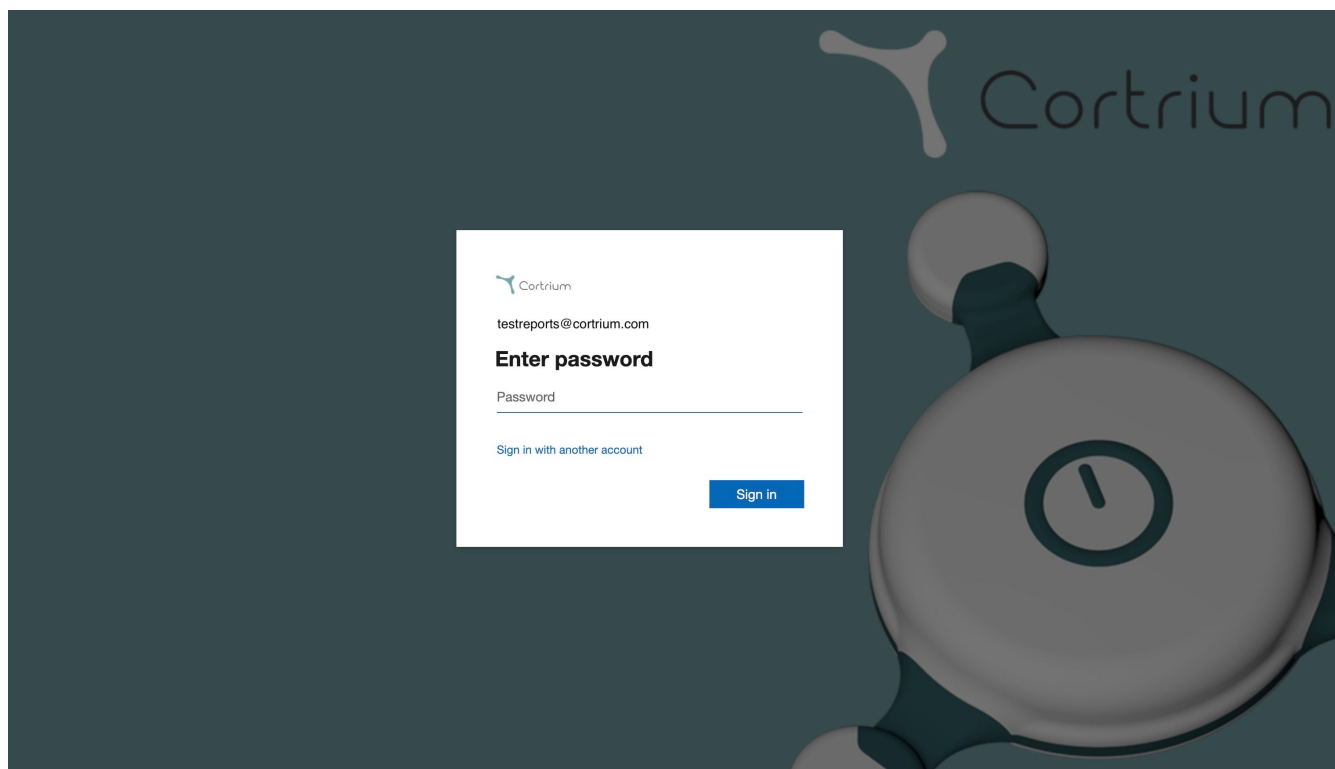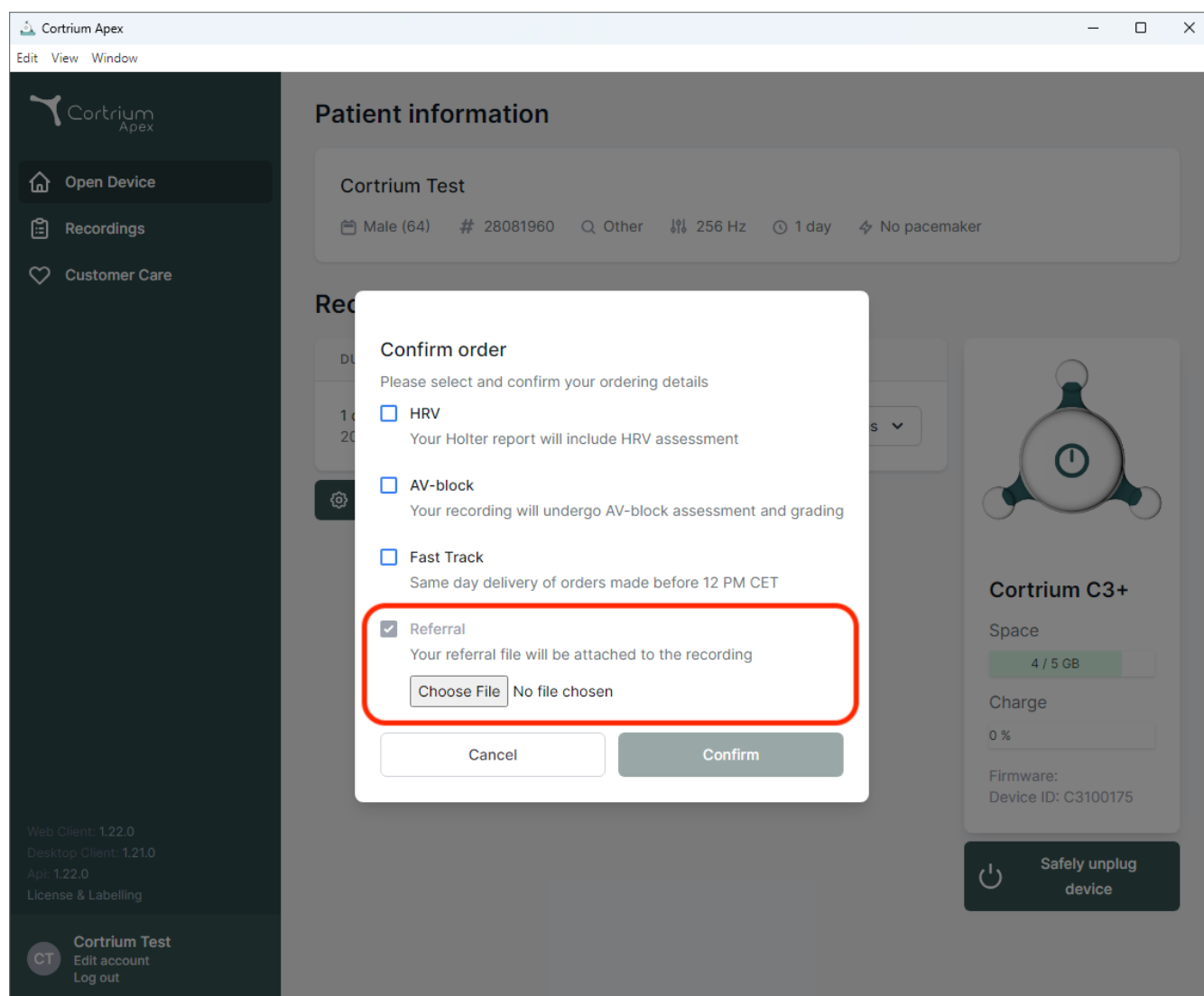
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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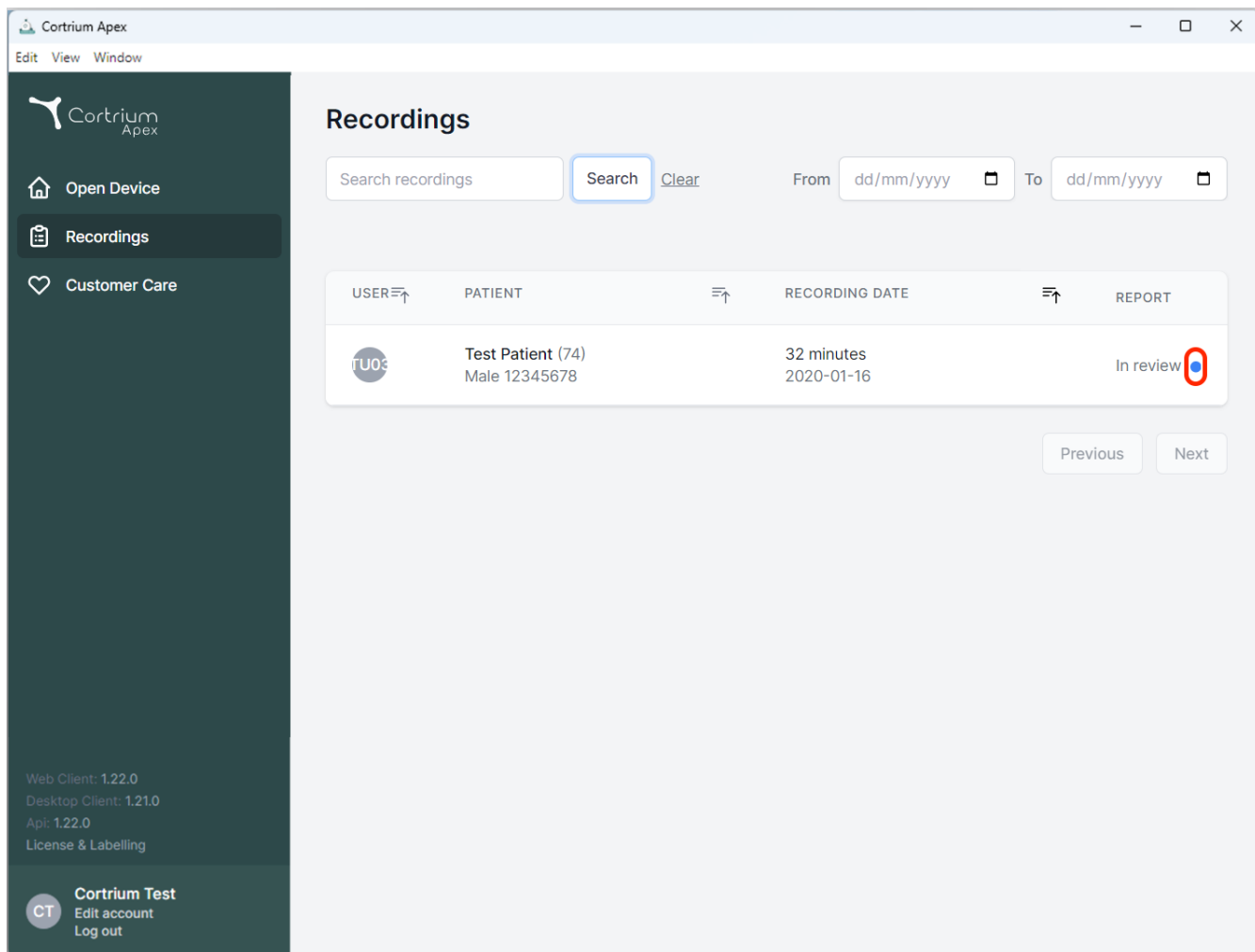
Search recordings [Clear](#)


From dd/mm/yyyy To dd/mm/yyyy

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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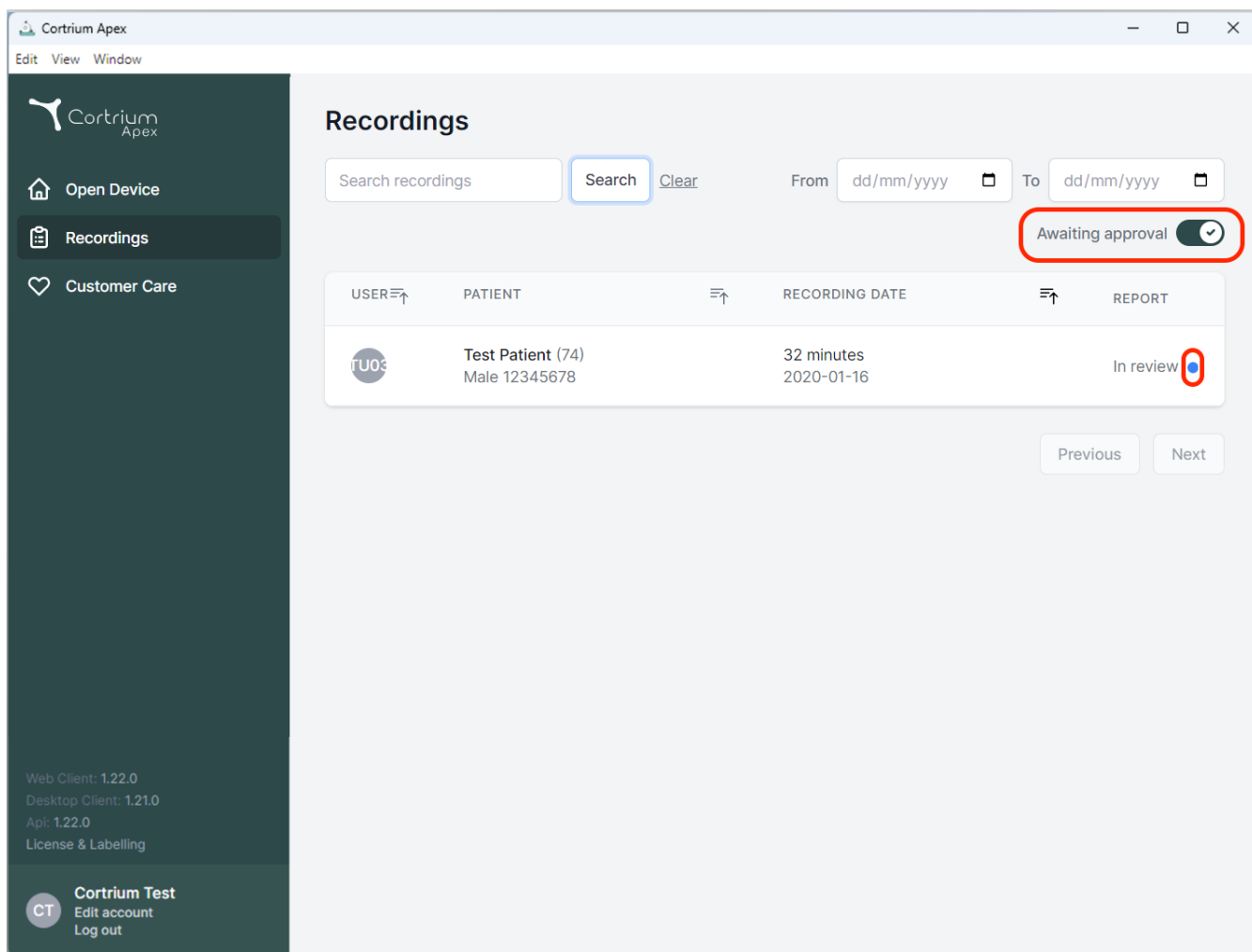
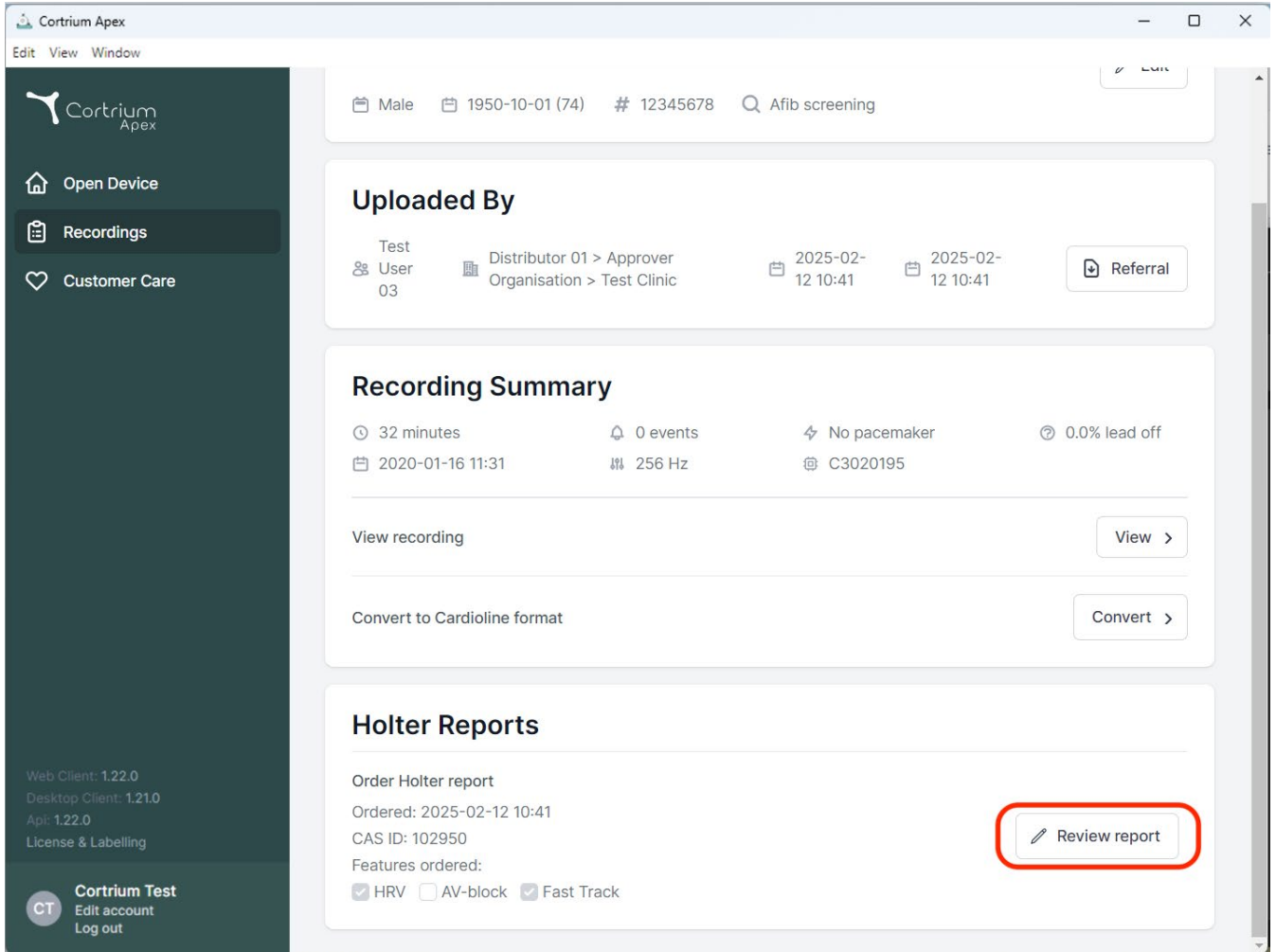
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with timestamps '2025-02-12 10:41' and '2025-02-12 10:41', and a 'Referral' button. The 'Recording Summary' section displays: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

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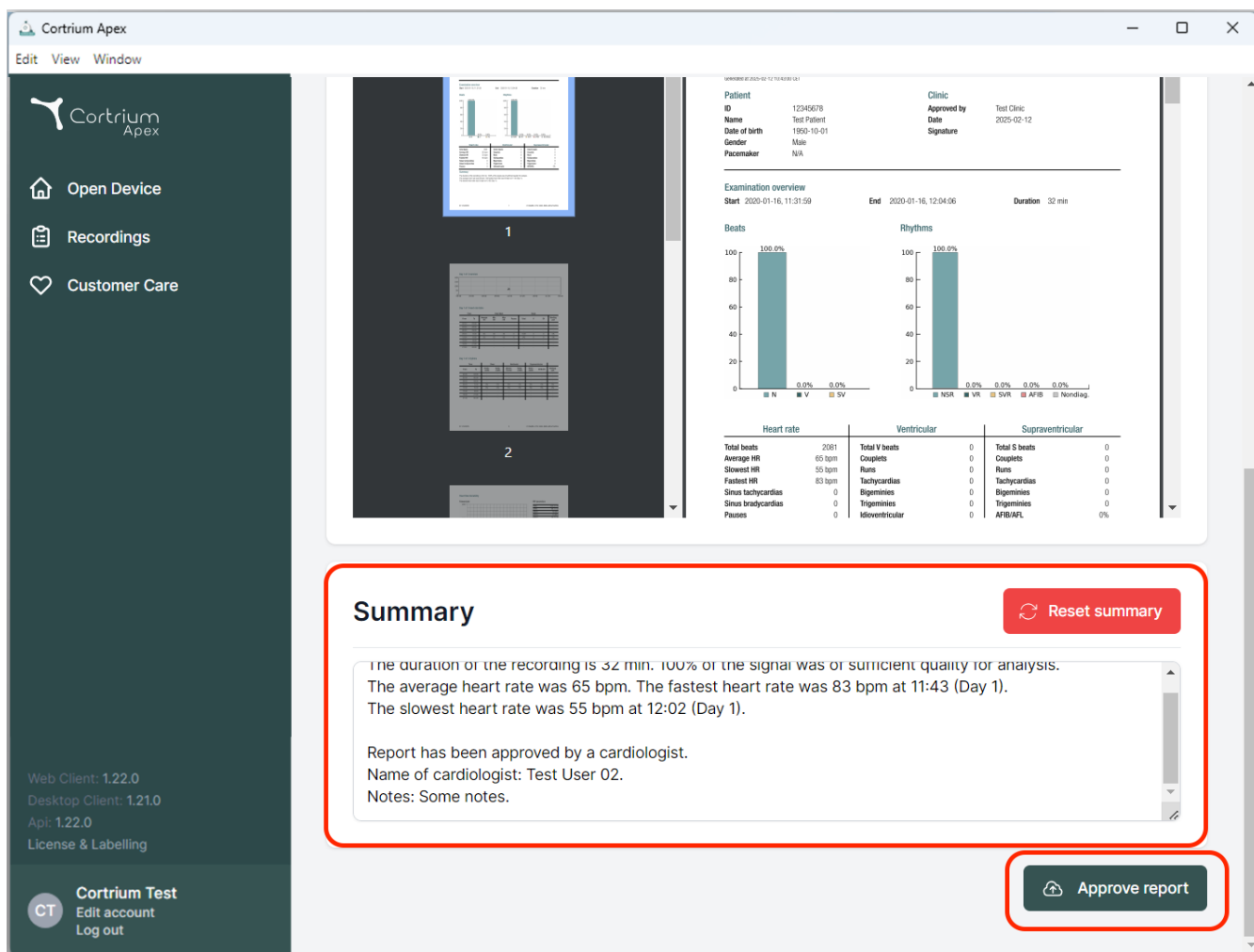Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

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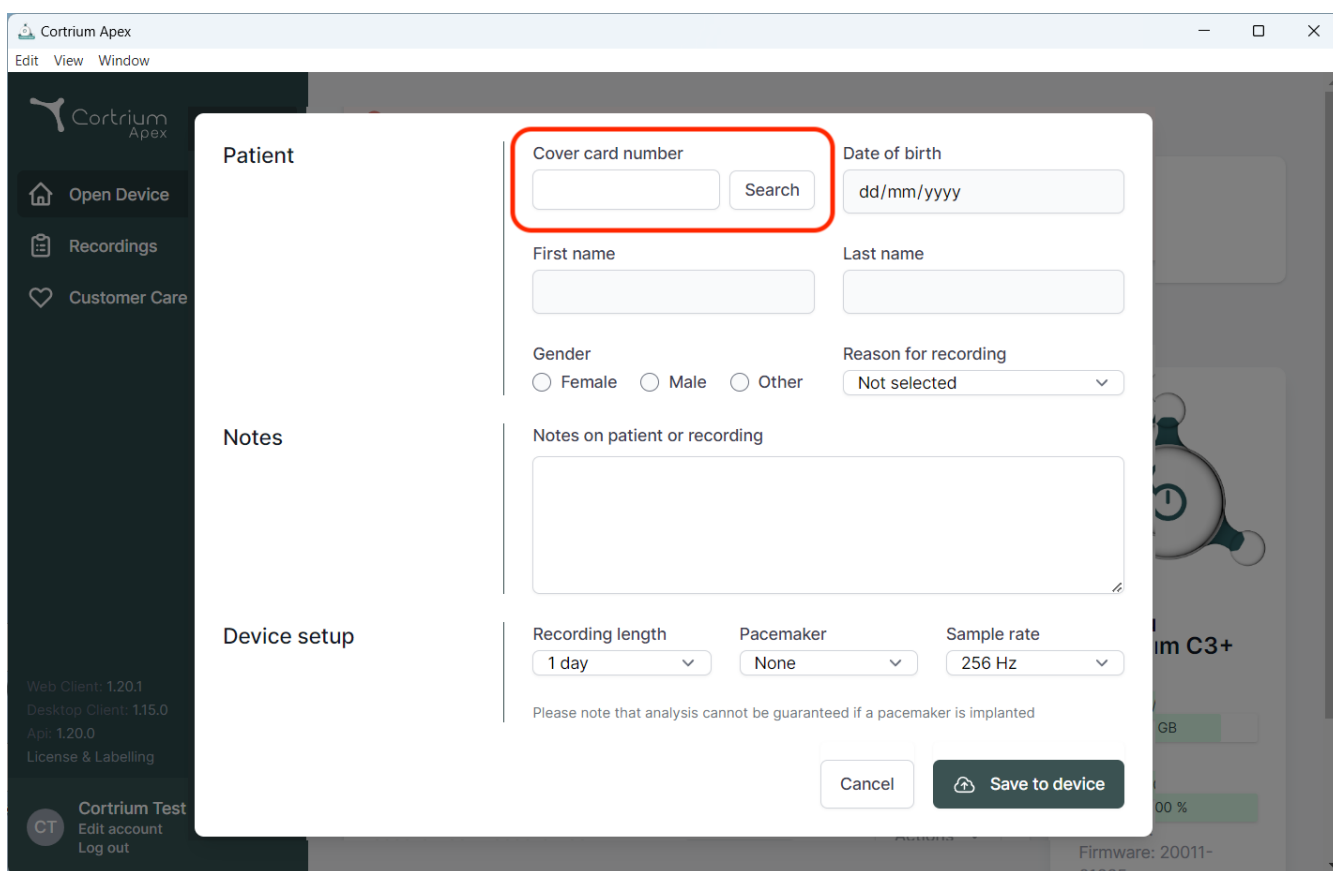
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for patient information. The form is divided into three main sections: "Patient", "Notes", and "Device setup".

Patient Section:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangular box.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu currently showing "1 day".
- Pacemaker:** A dropdown menu currently showing "None".
- Sample rate:** A dropdown menu currently showing "256 Hz".

Below the device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the form, there are two buttons: "Cancel" and "Save to device".

The background of the application shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar contains version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right corner of the sidebar shows a "Cortrium Test" button and links for "Edit account" and "Log out".

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