# Cortrium Apex 1.31

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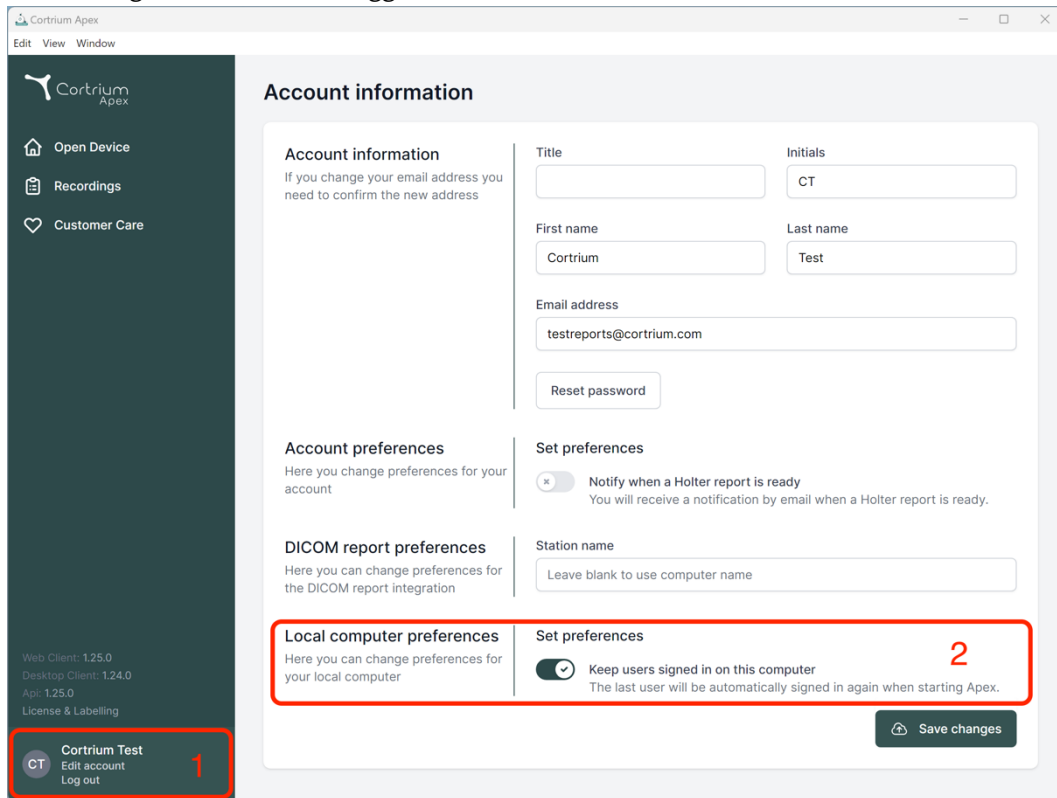
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client, Desktop Client, and API, along with a 'License & Labelling' link. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with options to 'Edit account' and 'Log out'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, First name, Last name, and Email address; 'Account preferences' with a toggle for notifications; 'DICOM report preferences' with a field for station name; and 'Local computer preferences' with a toggle for keeping users signed in. A red box highlights the 'Local computer preferences' section, and another red box highlights the 'Set preferences' section. A 'Save changes' button is located at the bottom right of the main content area.

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☒ **Notify when a Holter report is ready**  
You will receive a notification by email when a Holter report is ready.

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Station name:

**Local computer preferences**

Here you can change preferences for your local computer

☒ **Keep users signed in on this computer**  
The last user will be automatically signed in again when starting Apex.

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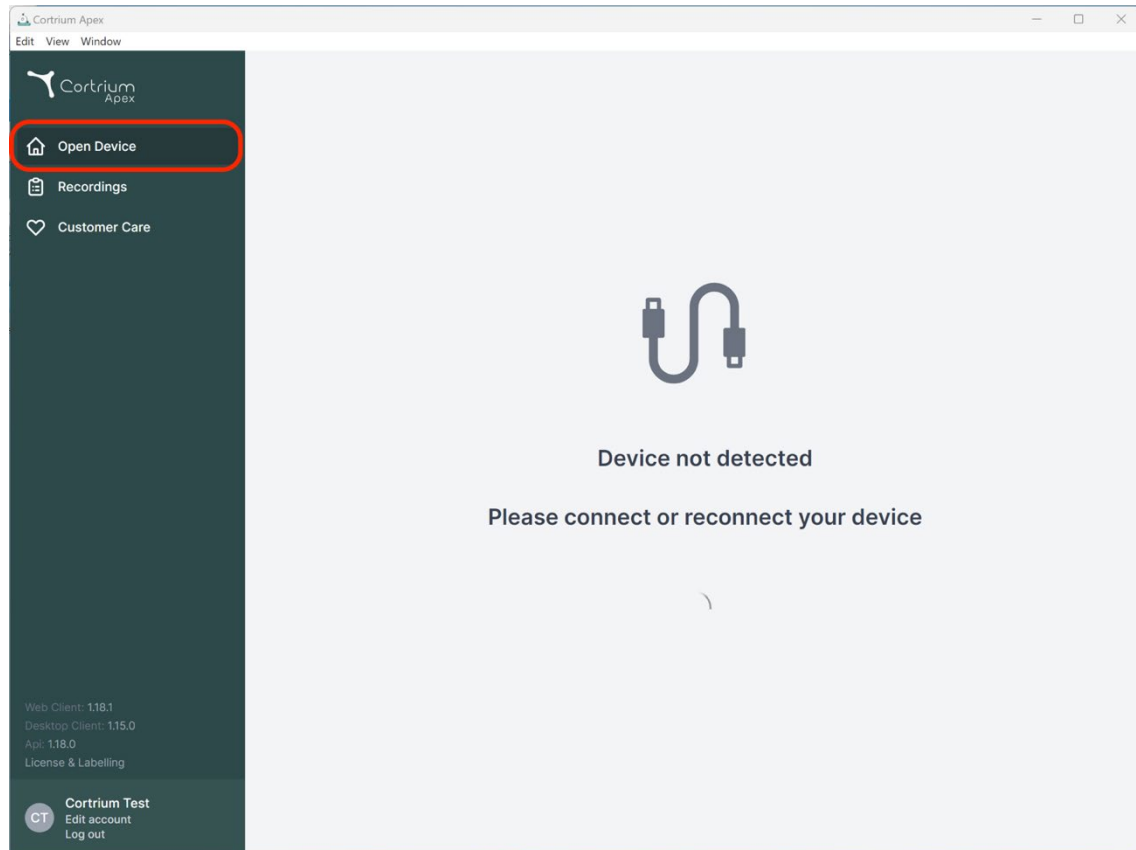
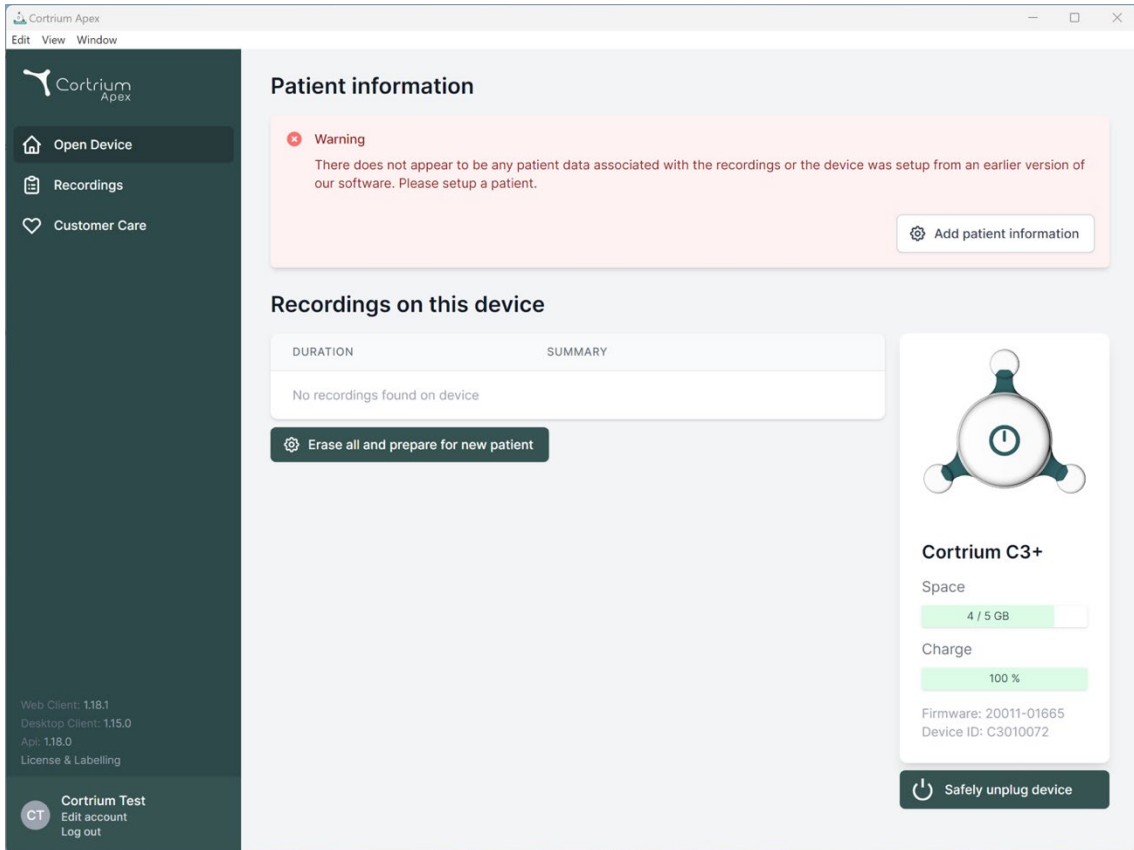
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area is titled 'Patient information' and features a red warning box stating: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table showing 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' displays 'Space' (4 / 5 GB) and 'Charge' (100 %) with progress bars, and includes firmware and device ID information. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex  
Edit View Window

**Patient information**

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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**Save to device**

**Warning** This will erase all recordings on the device

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
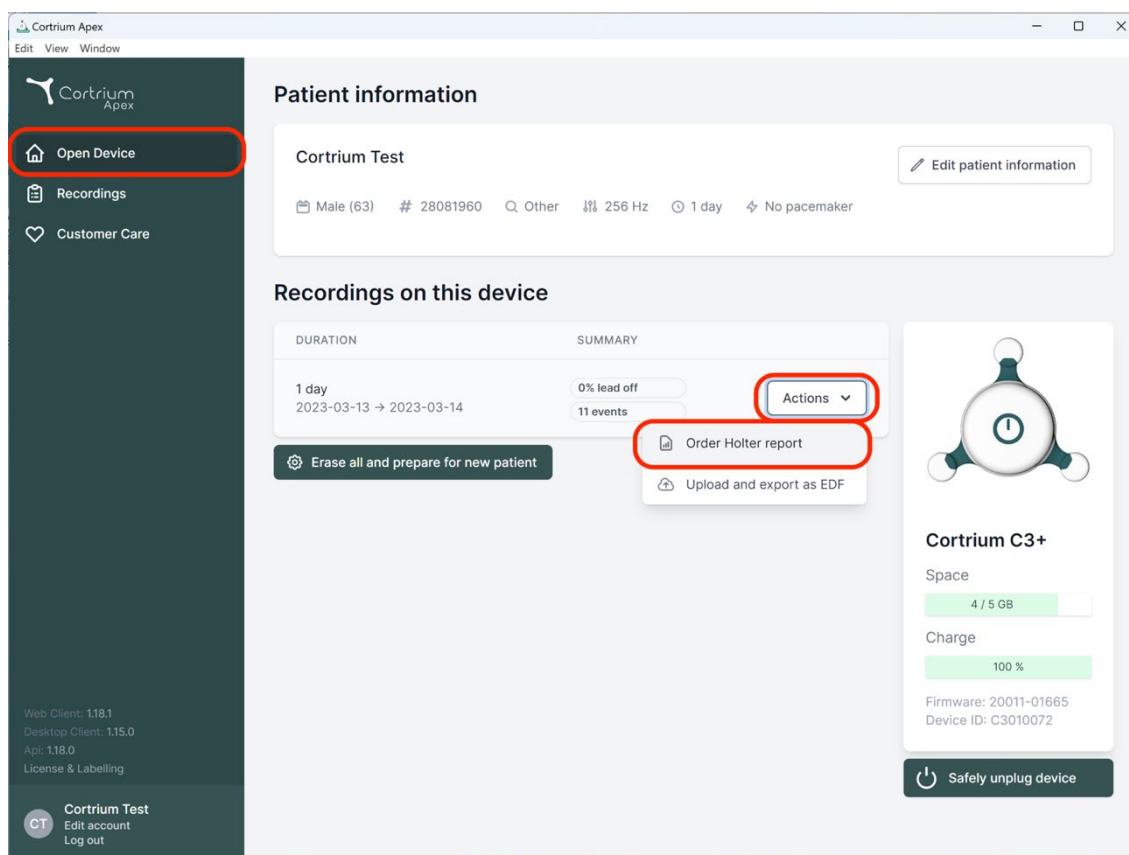
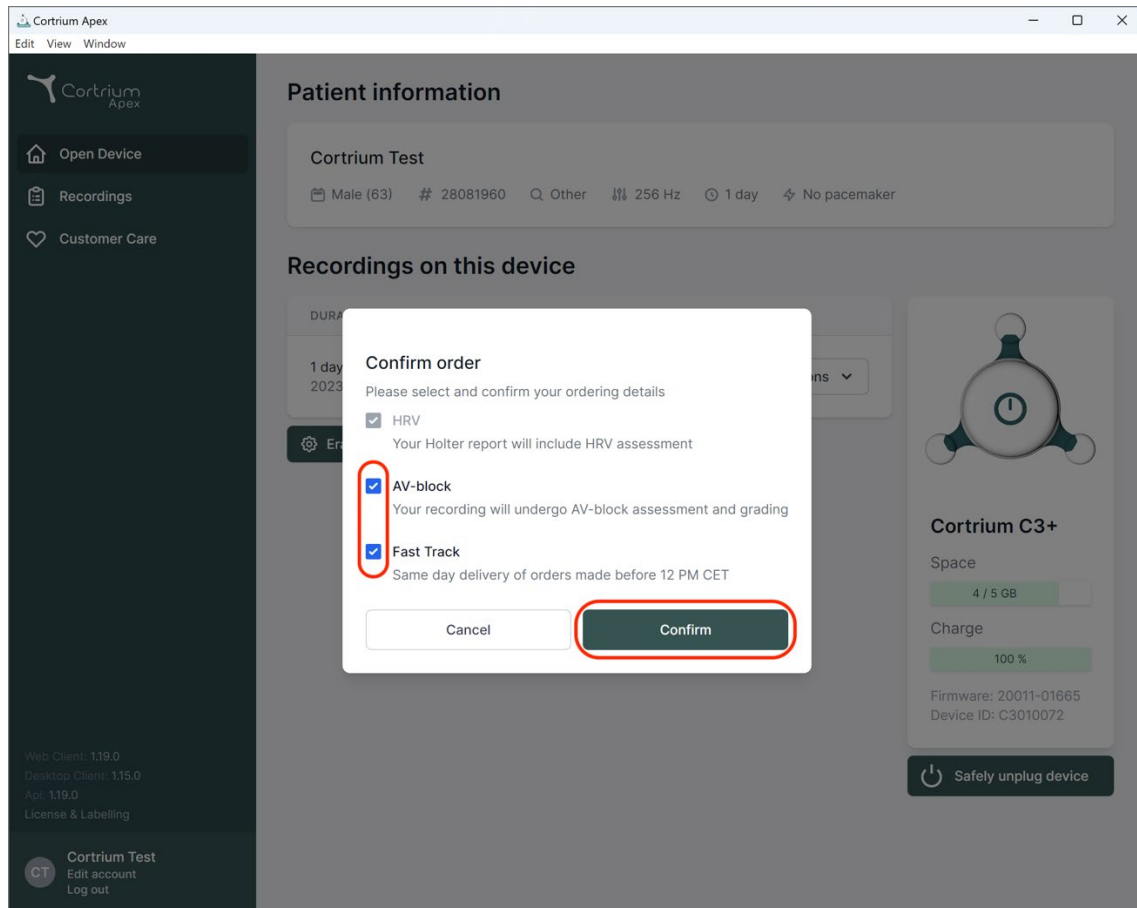
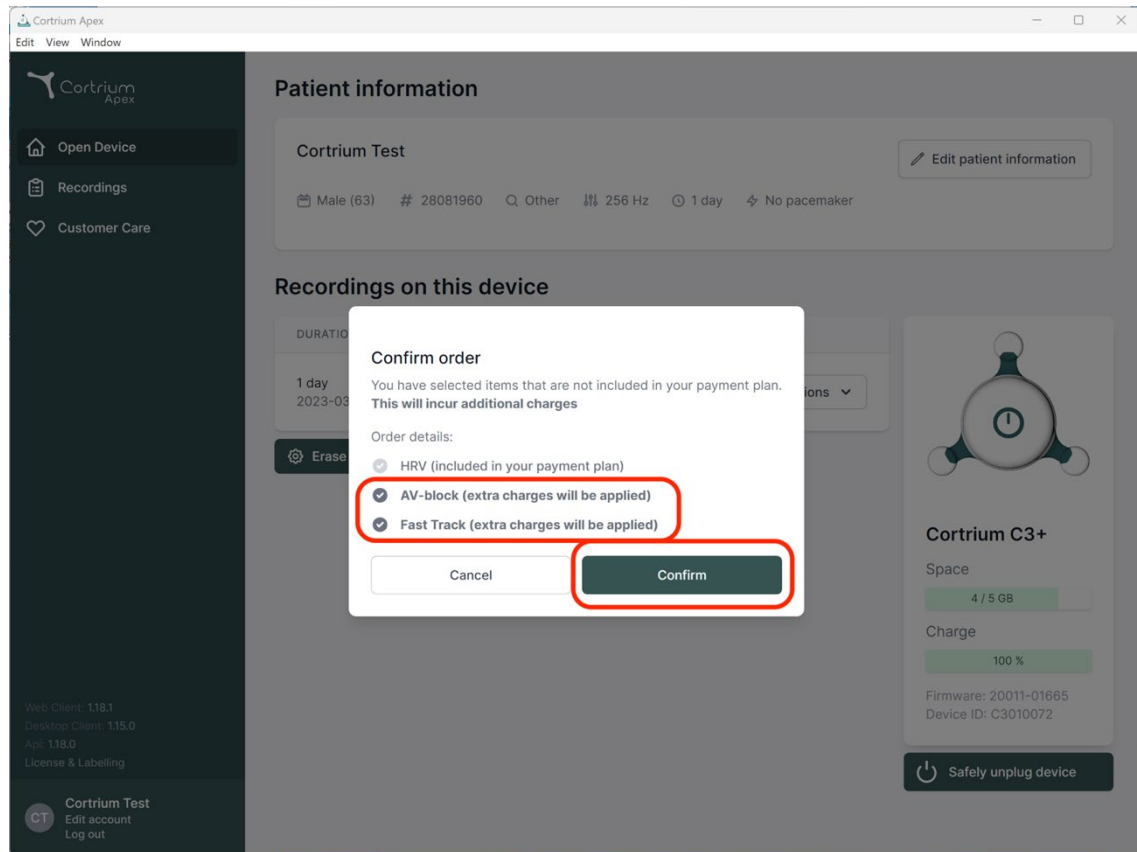
Herefter uploades optagelsen.

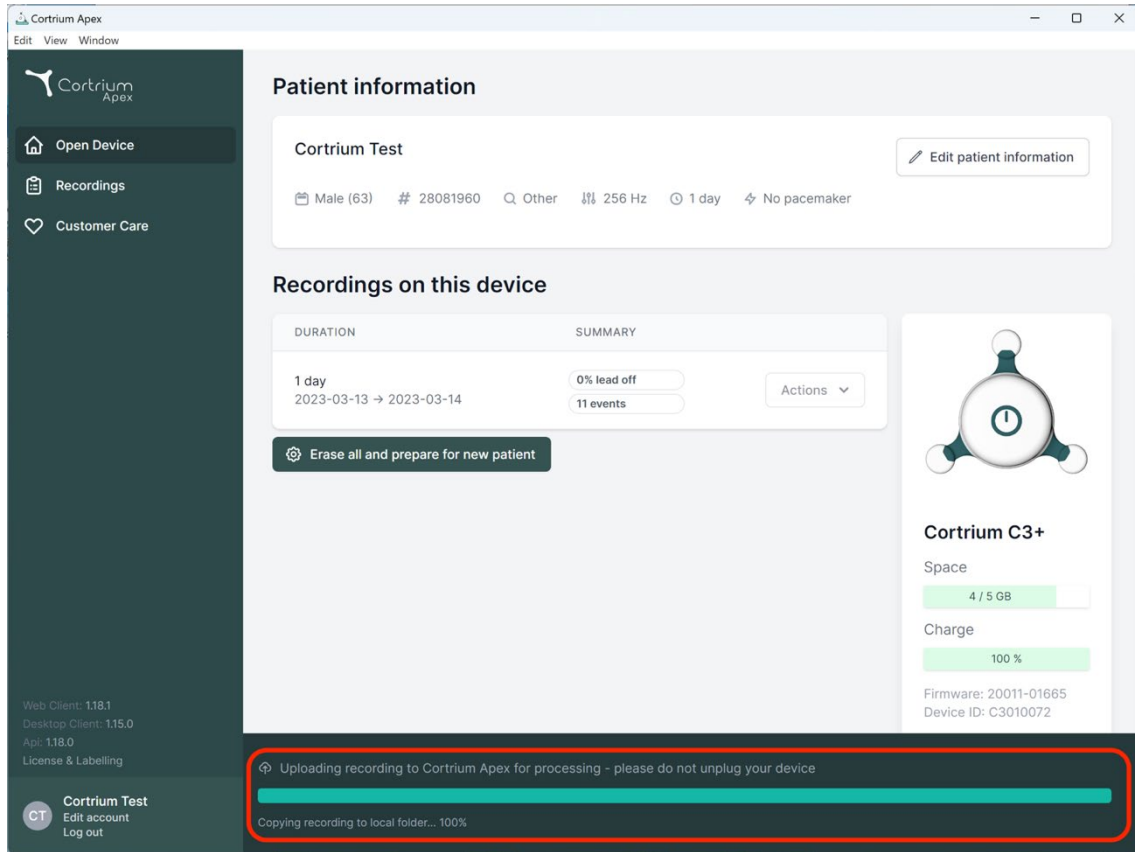The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'Api: 1.18.0', and 'License & Labelling'. At the bottom of the sidebar is a user profile for 'Cortrium Test' with an 'Edit account' and 'Log out' option. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and other parameters like '256 Hz' and '1 day'. A button 'Edit patient information' is present. Below this is a section 'Recordings on this device' with a table showing a recording for '1 day' (2023-03-13 to 2023-03-14) with '0% lead off' and '11 events'. An 'Actions' dropdown and an 'Erase all and prepare for new patient' button are also visible. On the right, a device icon is shown above 'Cortrium C3+' with status for 'Space' (4 / 5 GB) and 'Charge' (100 %). Firmware and device ID are listed at the bottom. A red box highlights a status bar at the bottom of the main area that reads: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'.

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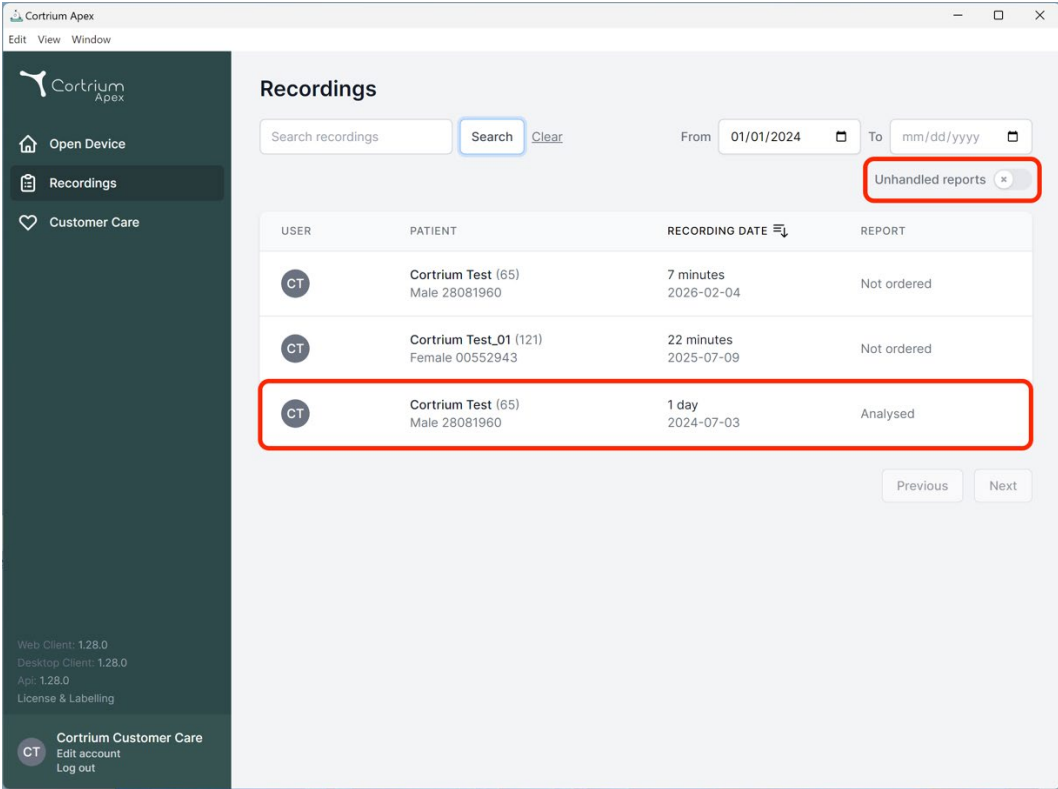
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main area is titled 'Recordings' and includes a search bar, a date range selector (From 01/01/2024 To mm/dd/yyyy), and a filter dropdown labeled 'Unhandled reports'. Below this is a table with the following data:

| USER | PATIENT                                   | RECORDING DATE           | REPORT      |
|------|-------------------------------------------|--------------------------|-------------|
| CT   | Cortrium Test (65)<br>Male 28081960       | 7 minutes<br>2026-02-04  | Not ordered |
| CT   | Cortrium Test_01 (121)<br>Female 00552943 | 22 minutes<br>2025-07-09 | Not ordered |
| CT   | Cortrium Test (65)<br>Male 28081960       | 1 day<br>2024-07-03      | Analysed    |

At the bottom of the table are 'Previous' and 'Next' buttons. The bottom of the sidebar shows version information (Web Client: 1.28.0, Desktop Client: 1.28.0, Api: 1.28.0) and a 'Cortrium Customer Care' section with links for 'Edit account' and 'Log out'.

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
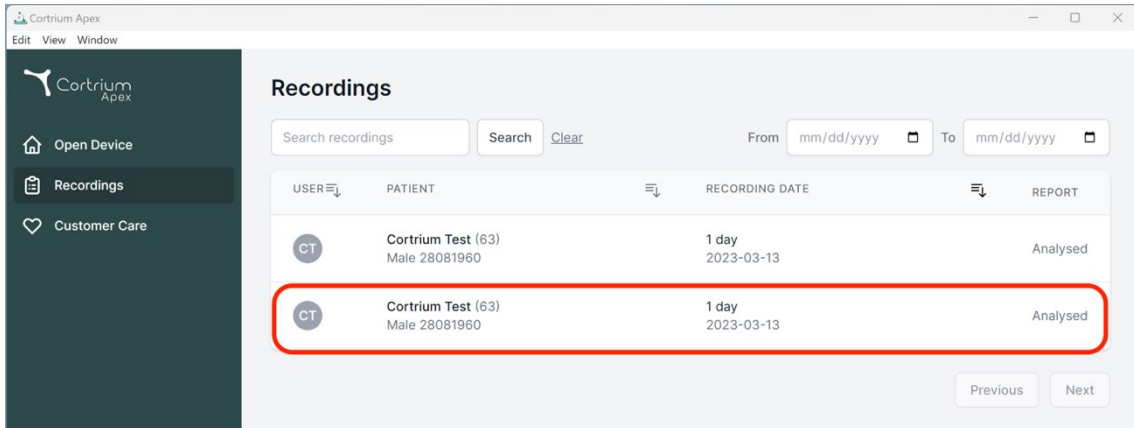
| USER | PATIENT                             | RECORDING DATE      | REPORT   |
|------|-------------------------------------|---------------------|----------|
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |

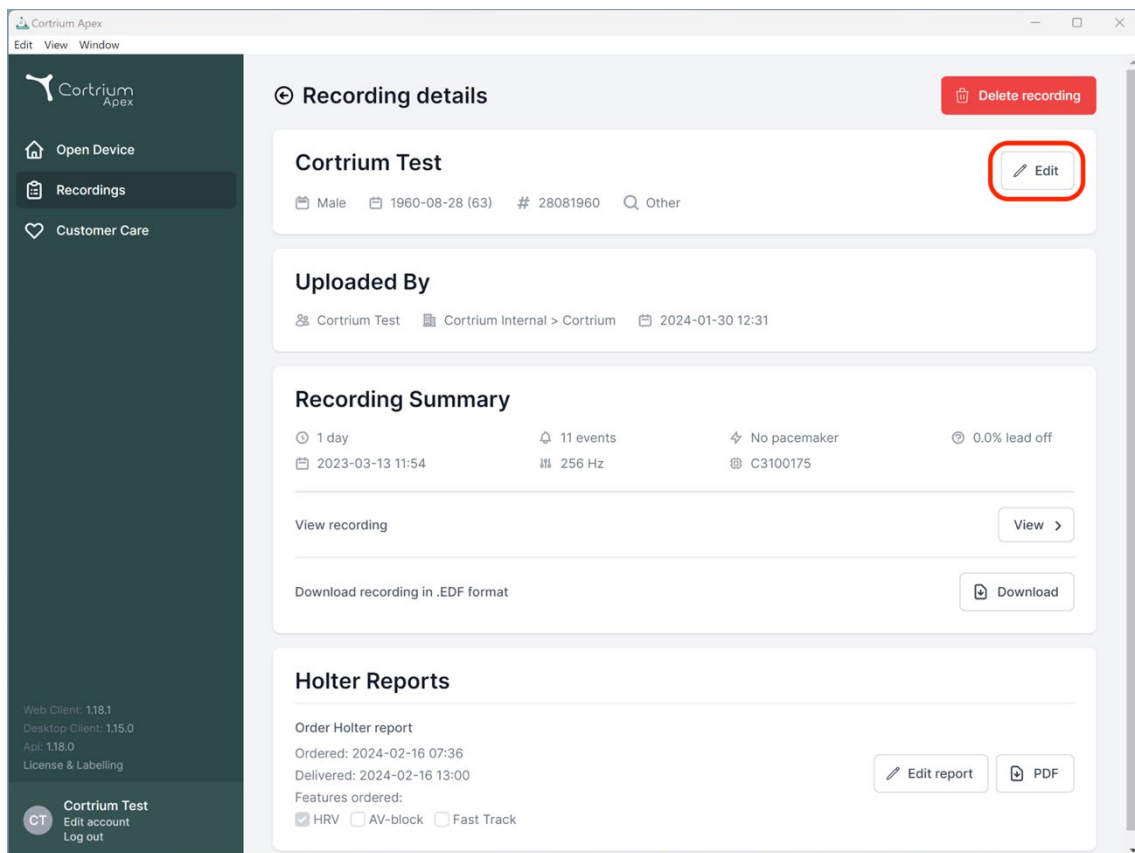Patientoplysningerne kan redigeres ved at klikke på knappen "Rediger".

**Recording details**

**Cortrium Test**

Male 1960-08-28 (63) # 28081960 Q Other

**Uploaded By**

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

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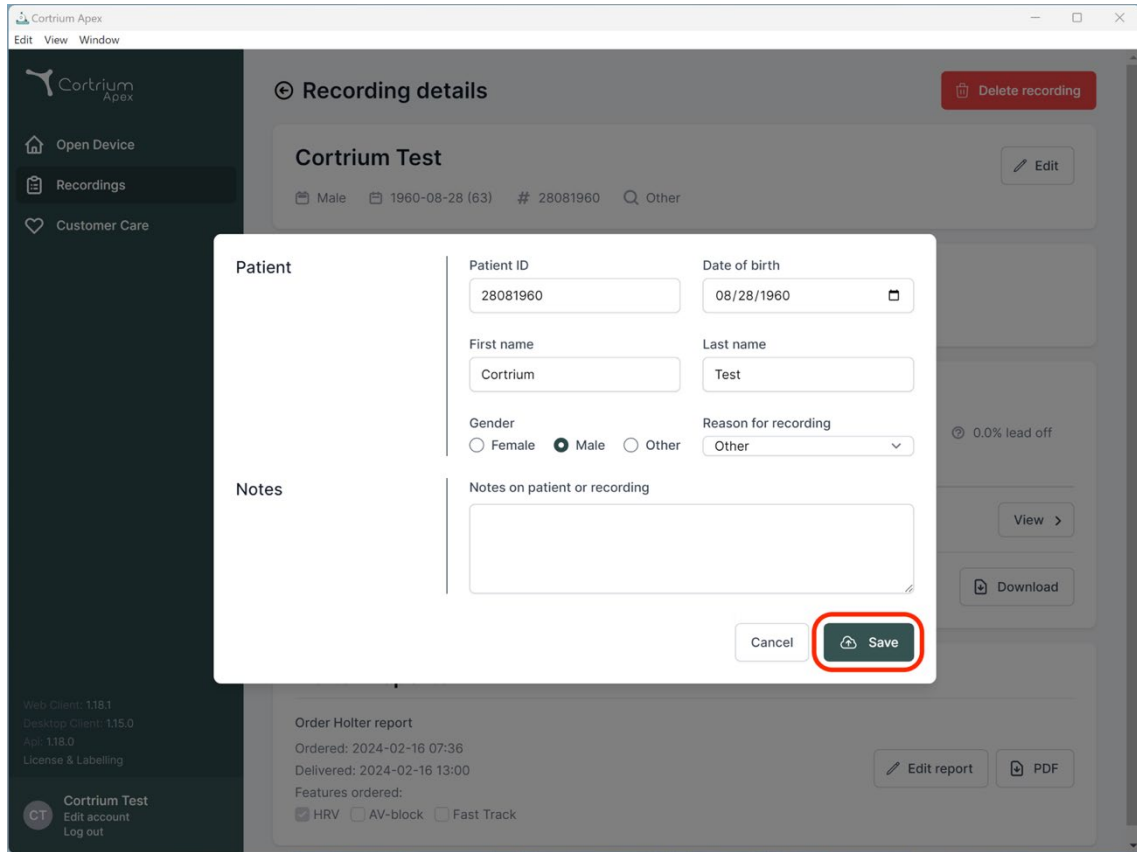
Order Holter report

Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
☒ HRV ☐ AV-block ☐ Fast Track

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
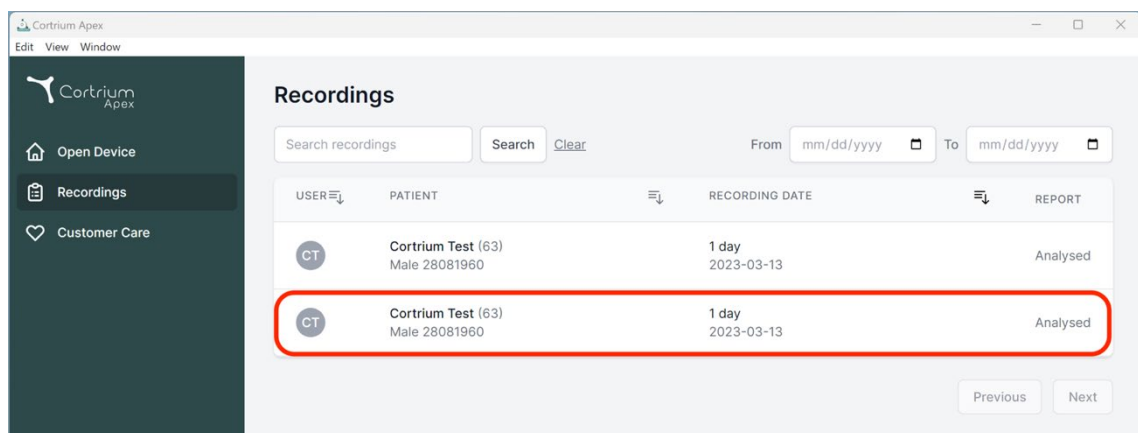
**Recordings**

Search recordings Search Clear From mm/dd/yyyy To mm/dd/yyyy

| USER | PATIENT                             | RECORDING DATE      | REPORT   |
|------|-------------------------------------|---------------------|----------|
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |

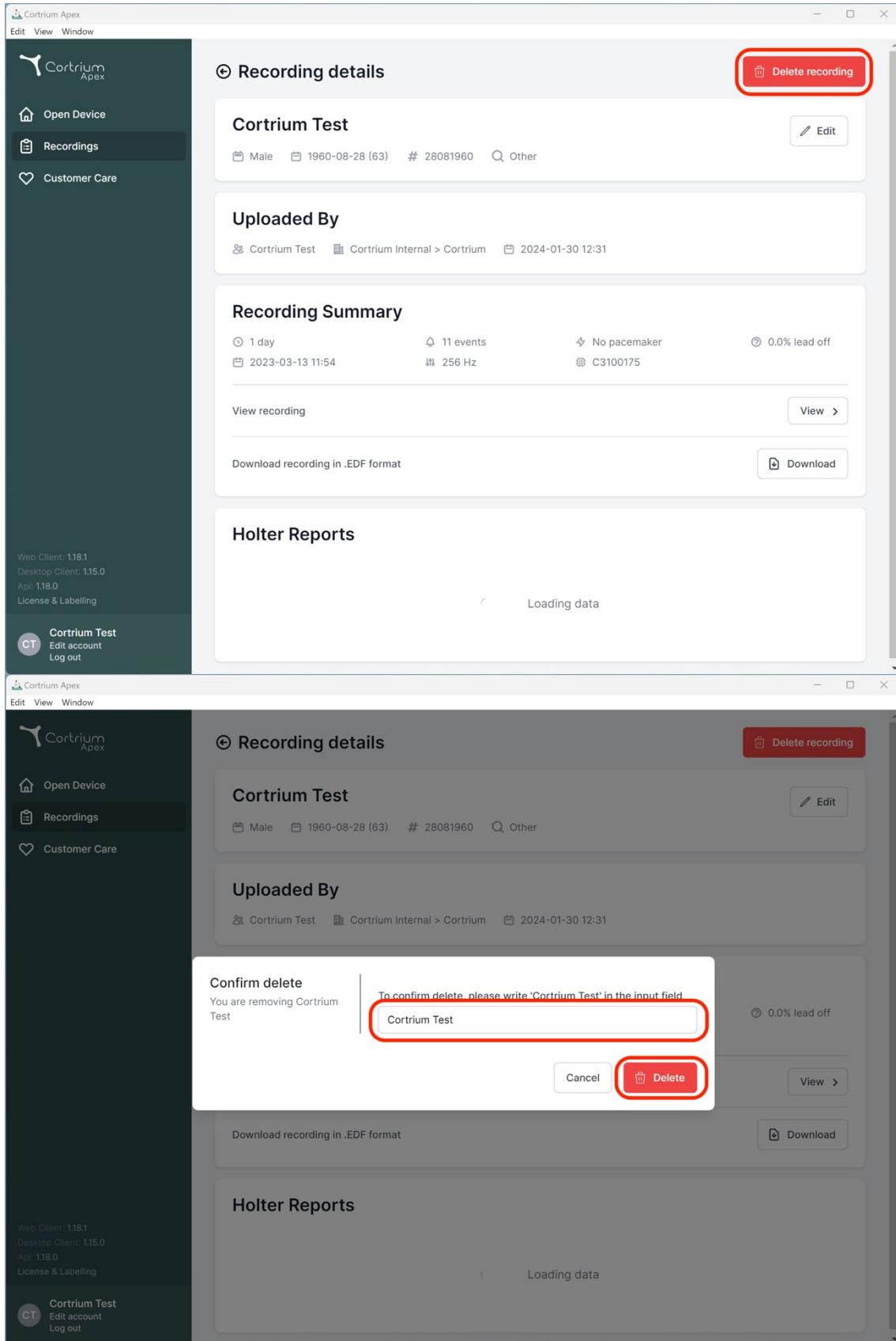Previous Next

**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

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**Holter Reports**

Loading data

**Confirm delete**  
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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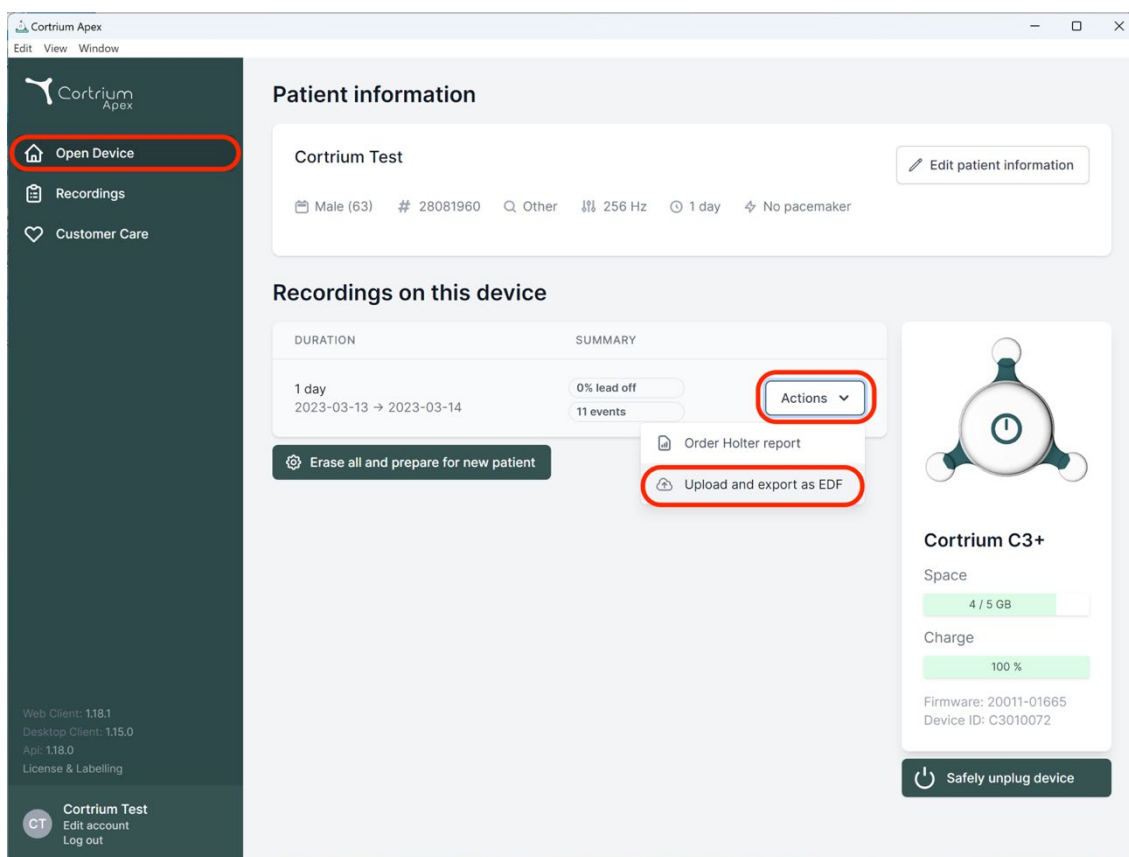
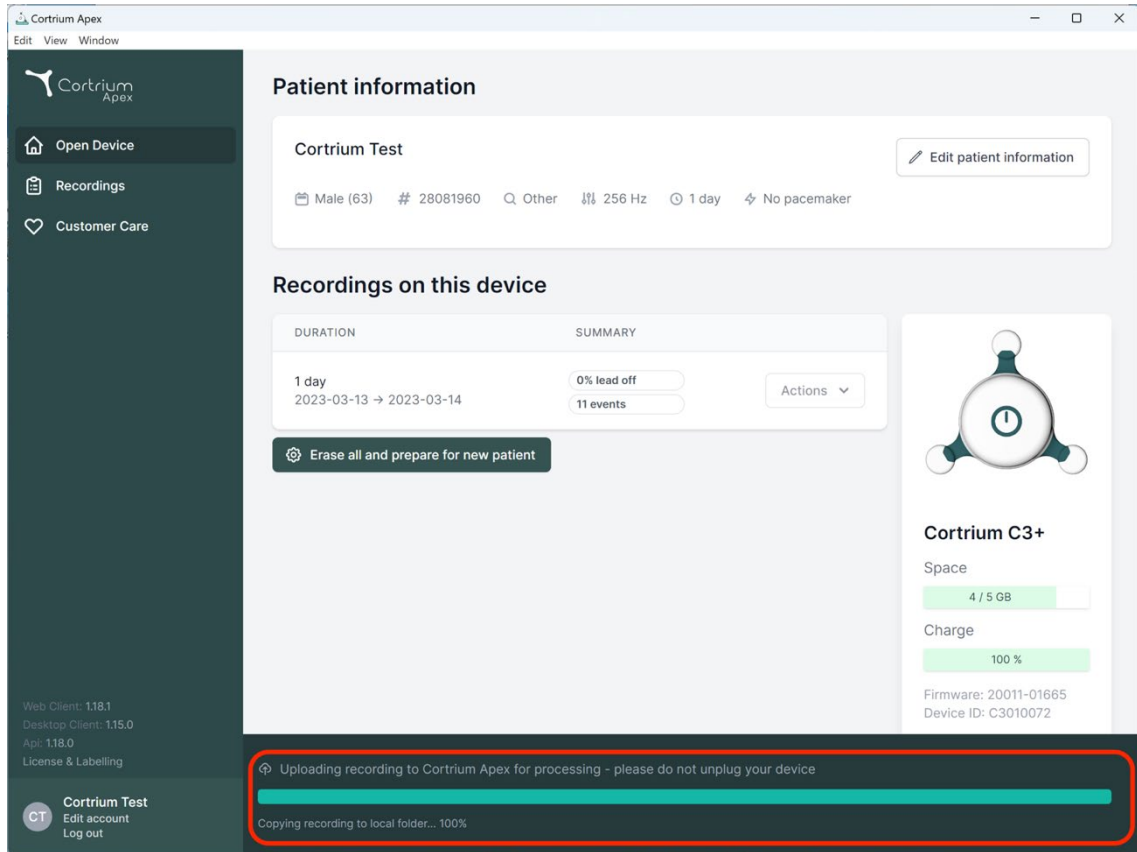
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  |                         |
|----------------------------------|--------------------------|-------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events | <a href="#">Actions</a> |

[Erase all and prepare for new patient](#)

**Cortrium C3+**

Space  
4 / 5 GB

Charge  
100 %

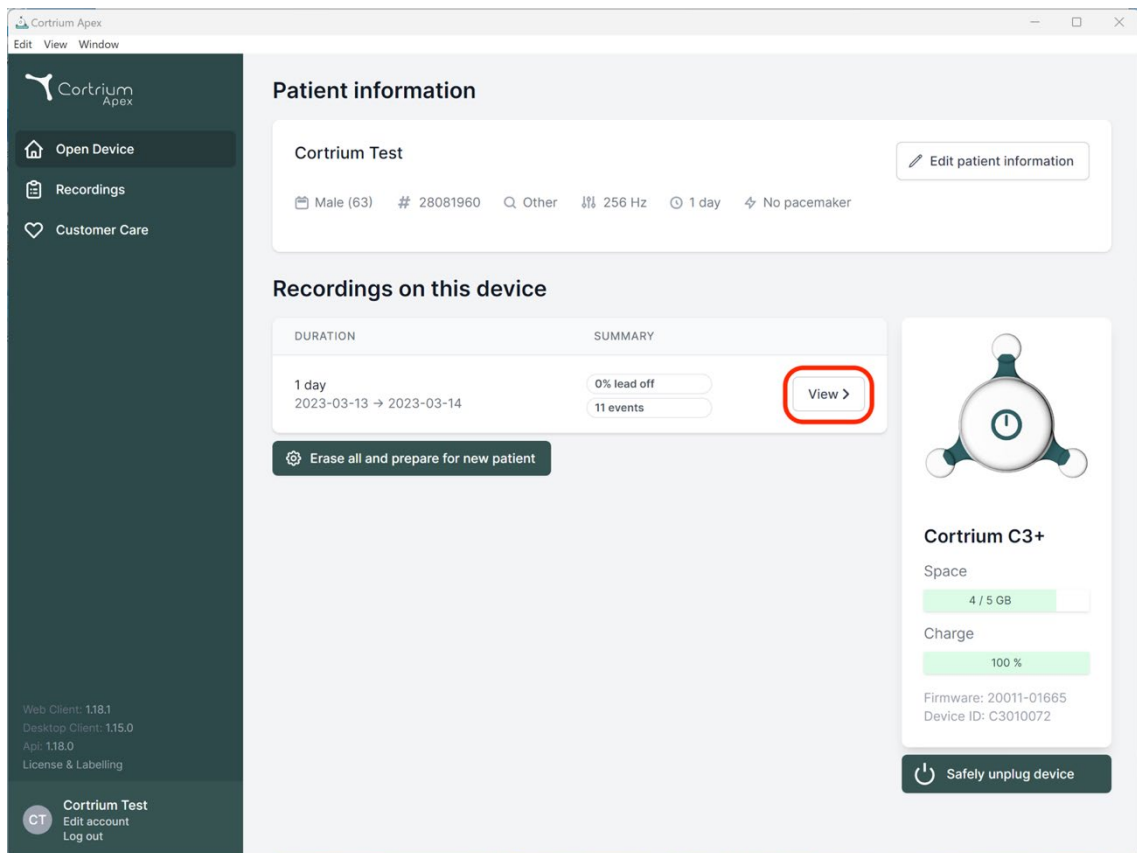
Firmware: 20011-01665  
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

**Cortrium Test**  
Edit account  
Log out

**Cortrium Apex**  
Edit View Window

**Patient information**

**Cortrium Test** [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  |                           |
|----------------------------------|--------------------------|---------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events | <a href="#">View &gt;</a> |

[Erase all and prepare for new patient](#)

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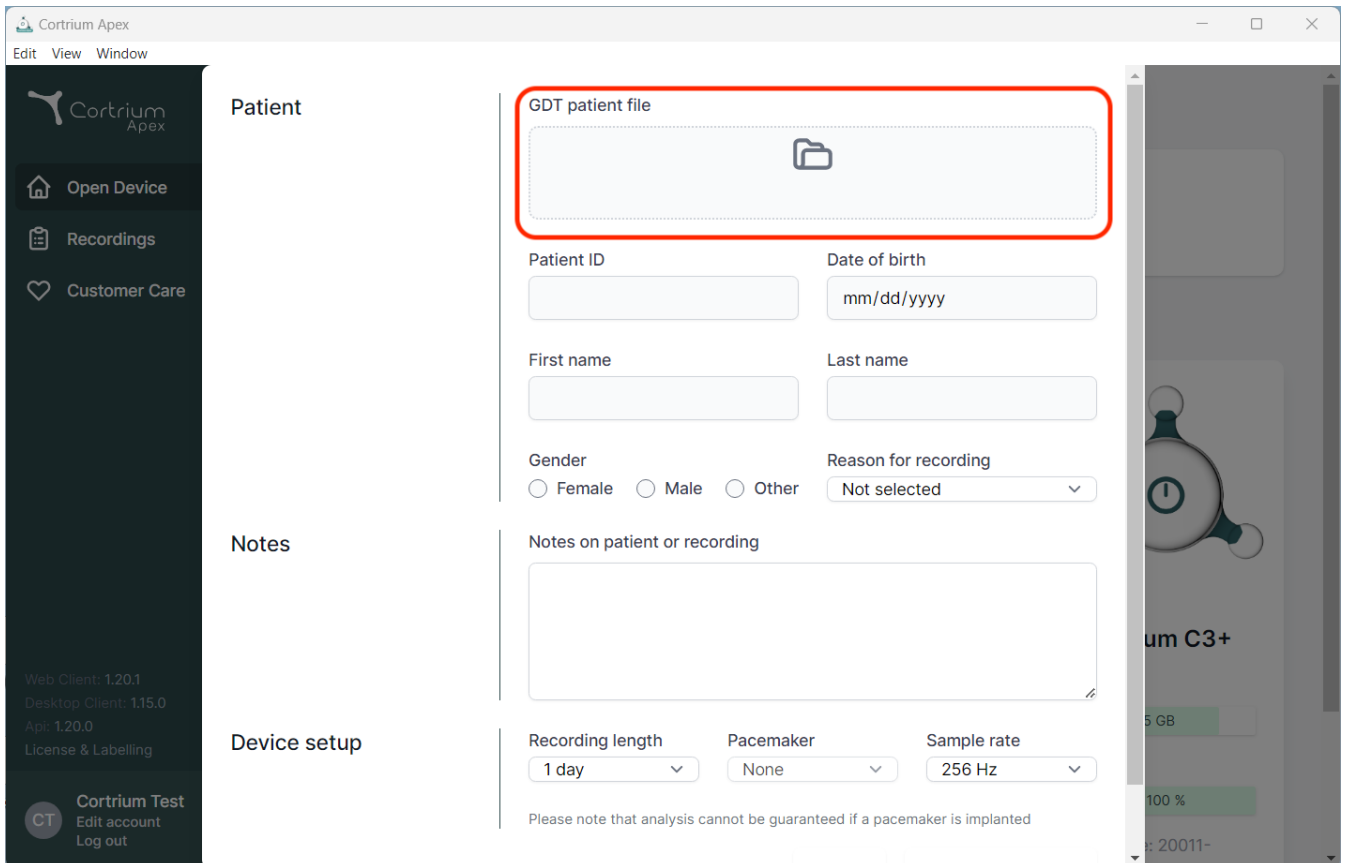
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section is highlighted and contains a form for entering patient information. The form includes a red-bordered box for the GDT patient file, followed by input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, and Last name. Below these are radio buttons for Gender (Female, Male, Other) and a dropdown menu for Reason for recording (Not selected). The Notes section has a text area for notes on the patient or recording. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Web Client: 1.20.1  
Desktop Client: 1.15.0  
Api: 1.20.0  
License & Labelling

Cortrium Test  
Edit account  
Log out

Patient

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None


Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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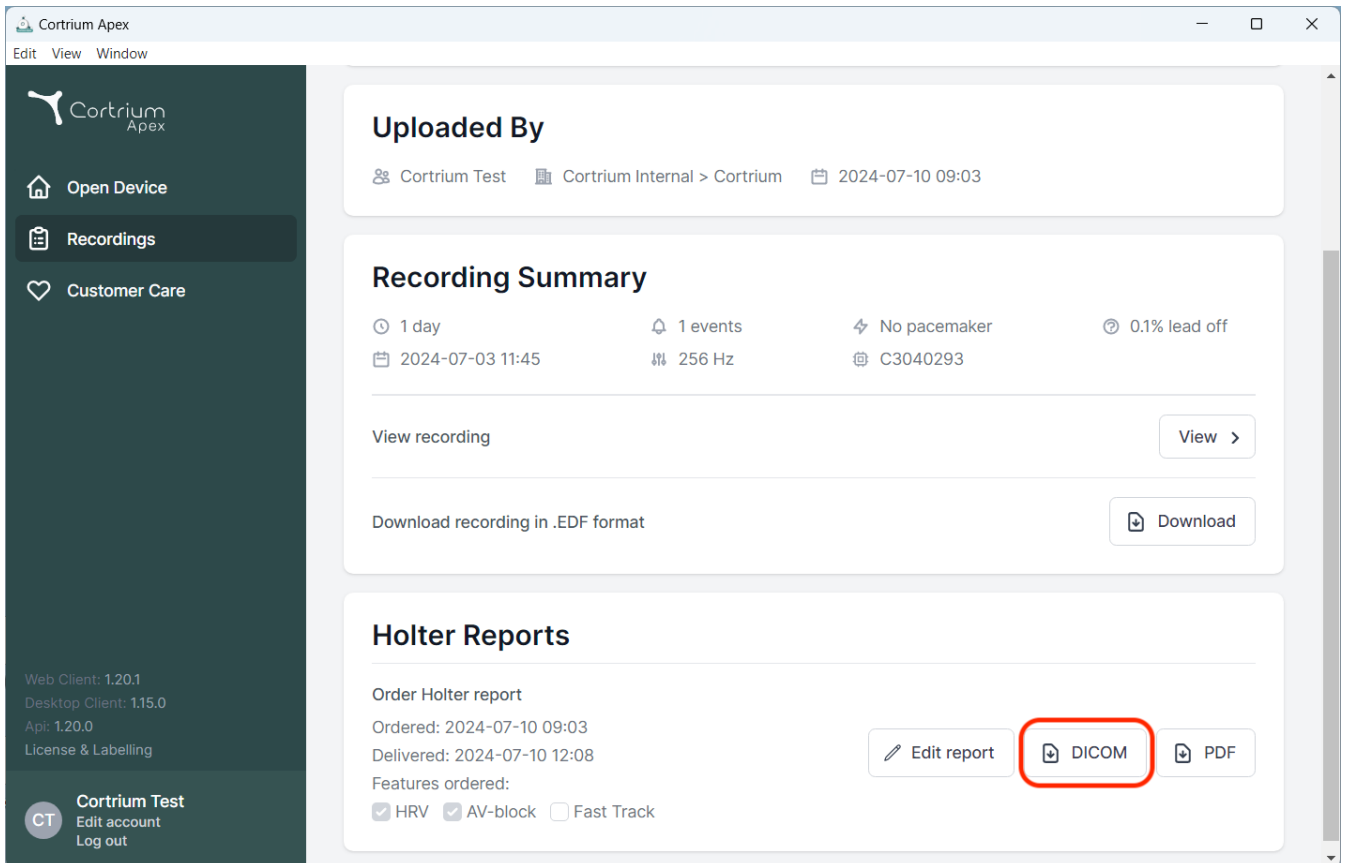The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By', 'Recording Summary', and 'Holter Reports'.

**Uploaded By**  
Cortrium Test   Cortrium Internal > Cortrium   2024-07-10 09:03

**Recording Summary**  
🕒 1 day   🔔 1 events   ⚡ No pacemaker   ⚙️ 0.1% lead off  
📅 2024-07-03 11:45   📡 256 Hz   📄 C3040293

View recording View >

Download recording in .EDF format Download

**Holter Reports**  
Order Holter report  
Ordered: 2024-07-10 09:03  
Delivered: 2024-07-10 12:08  
Features ordered:  
☒ HRV   ☒ AV-block   ☐ Fast Track

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## Account information

### Account information

If you change your email address you need to confirm the new address

Title

Initials

CT

First name

Cortrium

Last name

Test

Email address

testreports@cortrium.onmicrosoft.com

Reset password

### Account preferences

Here you change preferences for your account

### Set preferences

×

Notify when a Holter report is ready

You will receive a notification by email when a Holter report is ready.

### DICOM report preferences

Here you can change preferences for the DICOM report integration

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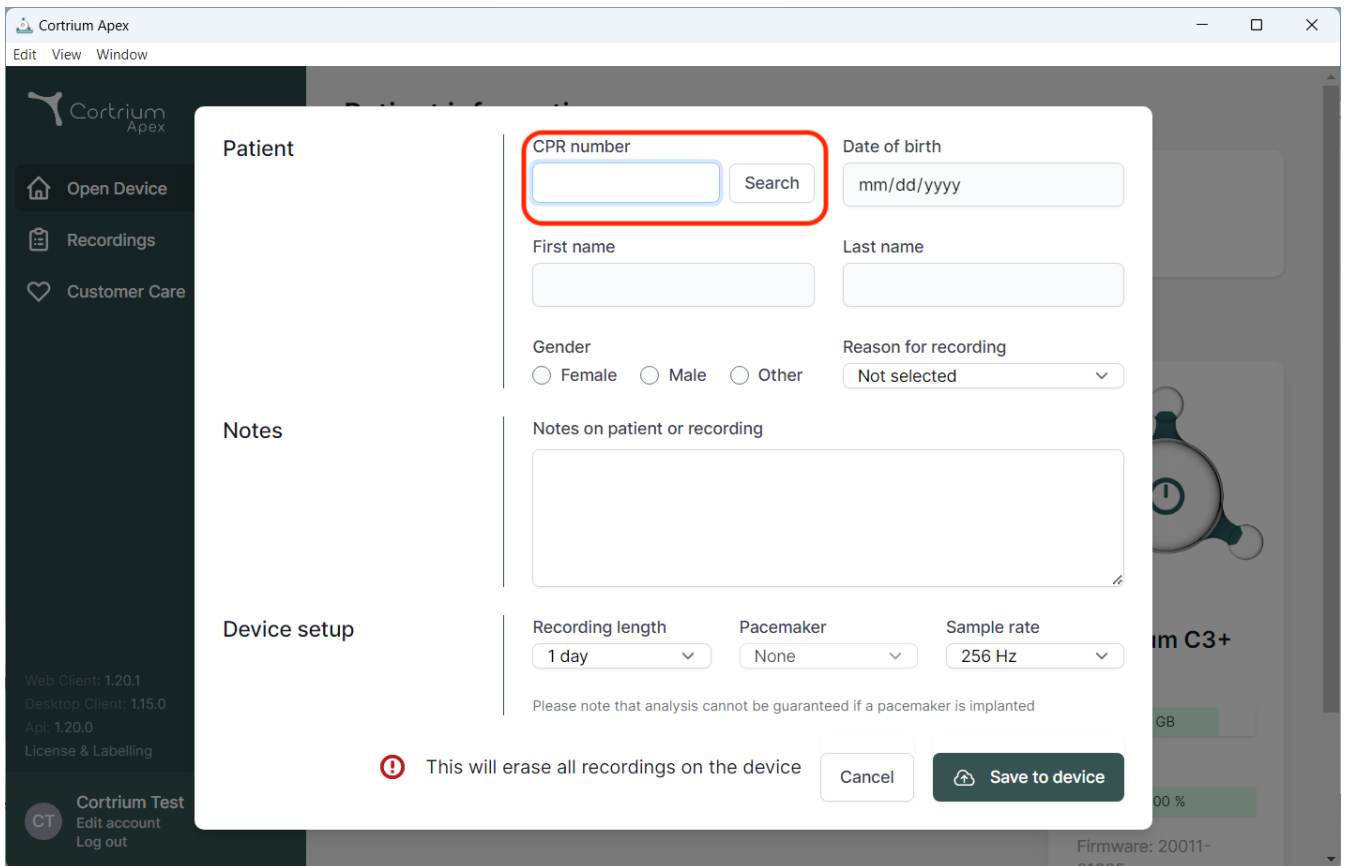
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for patient information. The form is divided into sections: "Patient", "Notes", and "Device setup".

**Patient section:**

- CPR number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangular box.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Three radio buttons labeled "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with the option "Not selected".

**Notes section:**

- Notes on patient or recording:** A large text area for entering notes.

**Device setup section:**

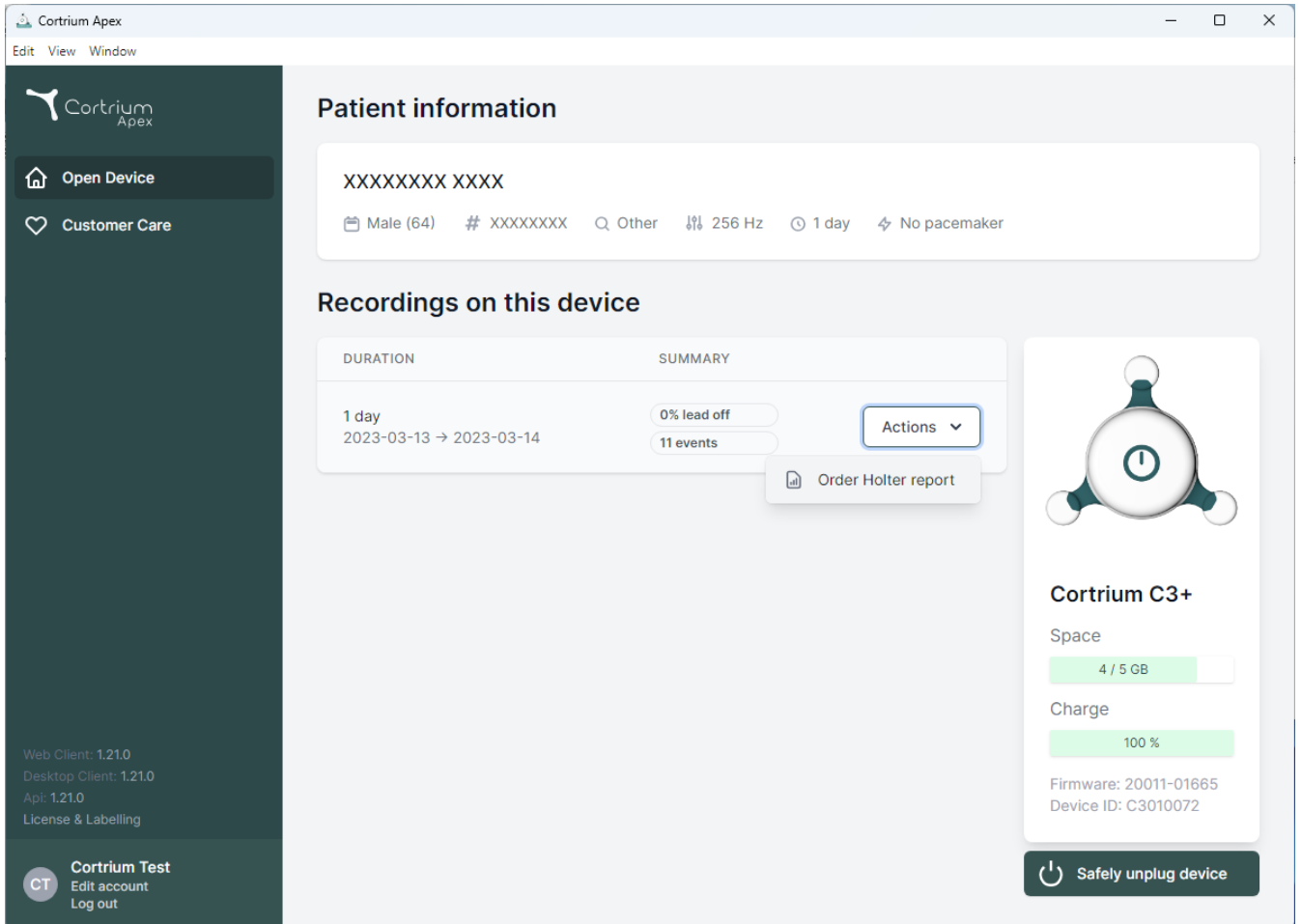
- Recording length:** A dropdown menu with the option "1 day".
- Pacemaker:** A dropdown menu with the option "None".
- Sample rate:** A dropdown menu with the option "256 Hz".

Below the device setup section, there is a warning message: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom of the modal, there is a red warning icon and the text: "This will erase all recordings on the device". To the right of this message are two buttons: "Cancel" and "Save to device".

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device" and "Customer Care". The main content area is divided into two sections: "Patient information" and "Recordings on this device".

**Patient information**

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  |
|----------------------------------|--------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events |

Actions

Order Holter report

**Cortrium C3+**

Space  
4 / 5 GB

Charge  
100 %

Firmware: 20011-01665  
Device ID: C3010072

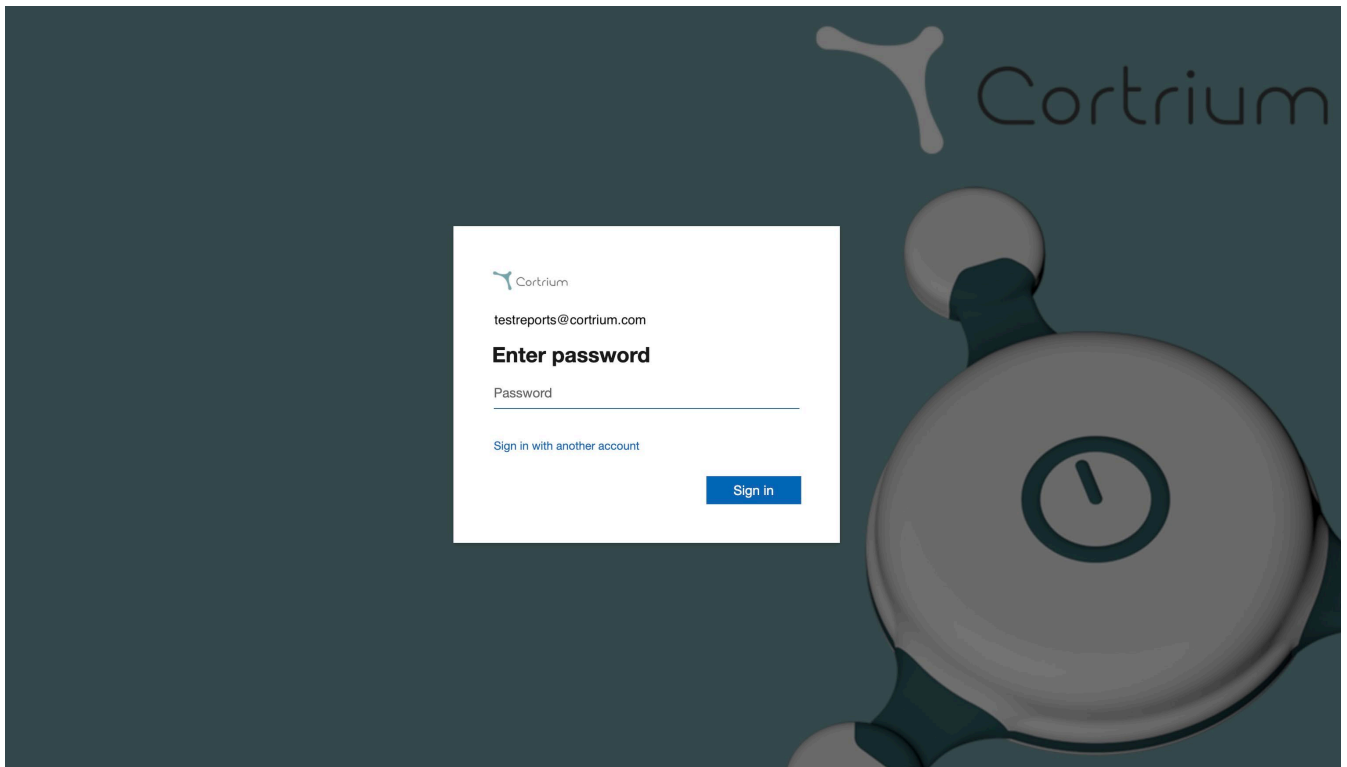
Safely unplug device

Web Client: 1.21.0  
Desktop Client: 1.21.0  
Api: 1.21.0  
License & Labelling

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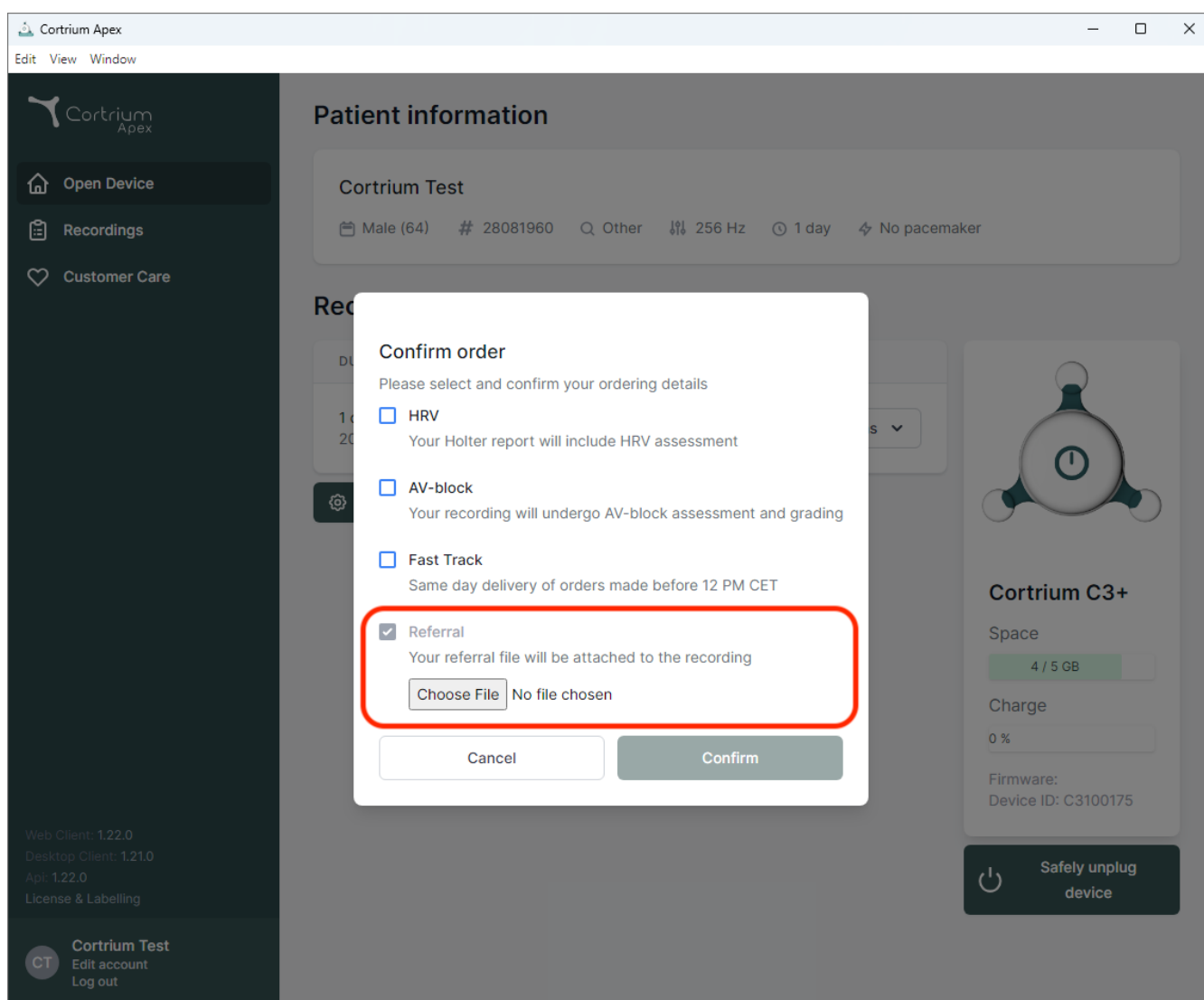
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
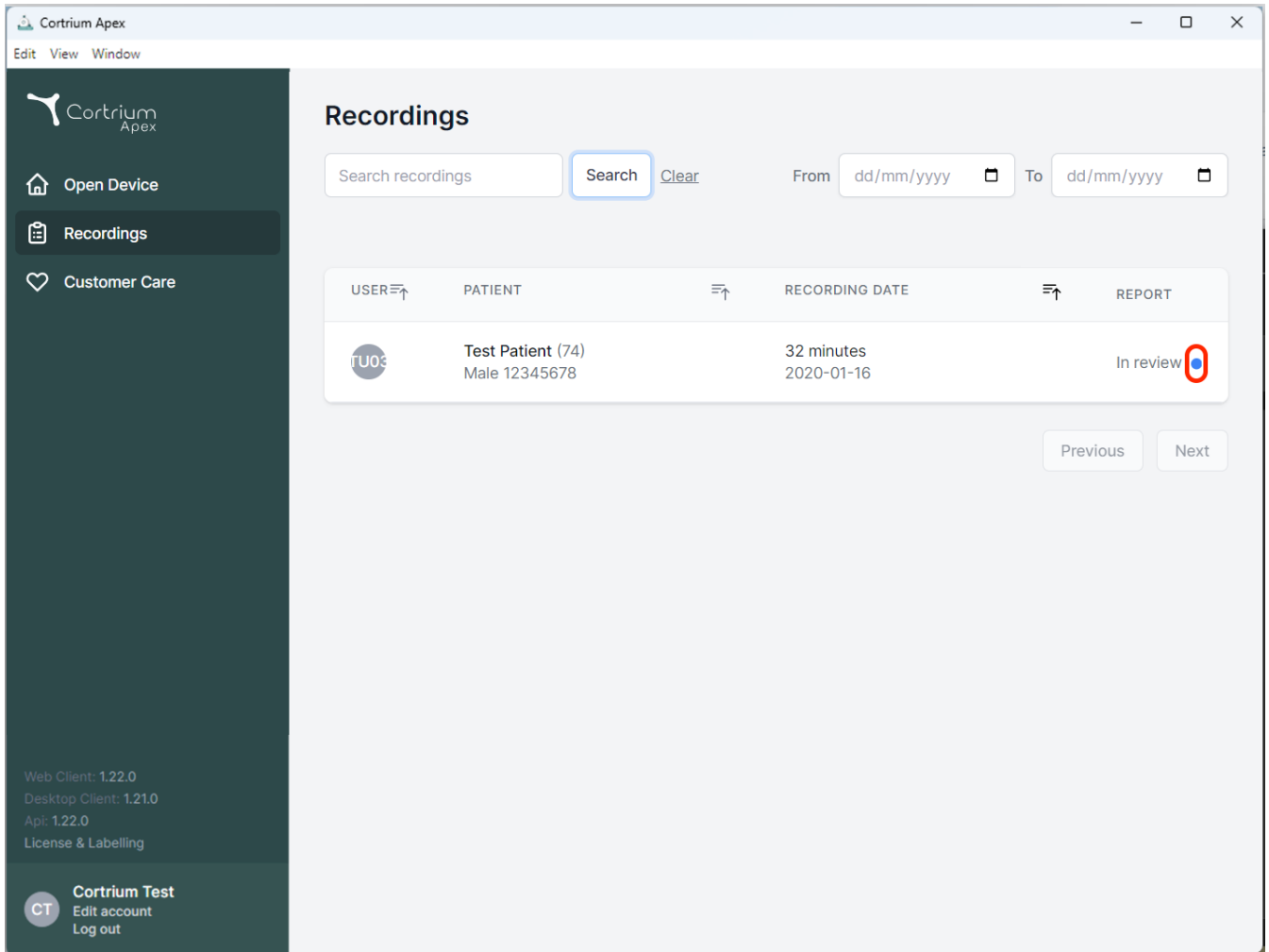
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (selected), and "Customer Care". The main area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. A single recording is listed for "Test Patient (74)" with a duration of "32 minutes" and a date of "2020-01-16". The "REPORT" column shows "In review" with a blue dot icon circled in red. At the bottom right of the table are "Previous" and "Next" buttons. The footer of the sidebar displays version information and a user profile for "Cortrium Test".

| USER | PATIENT                            | RECORDING DATE           | REPORT                                                                                        |
|------|------------------------------------|--------------------------|-----------------------------------------------------------------------------------------------|
| TU03 | Test Patient (74)<br>Male 12345678 | 32 minutes<br>2020-01-16 | In review  |

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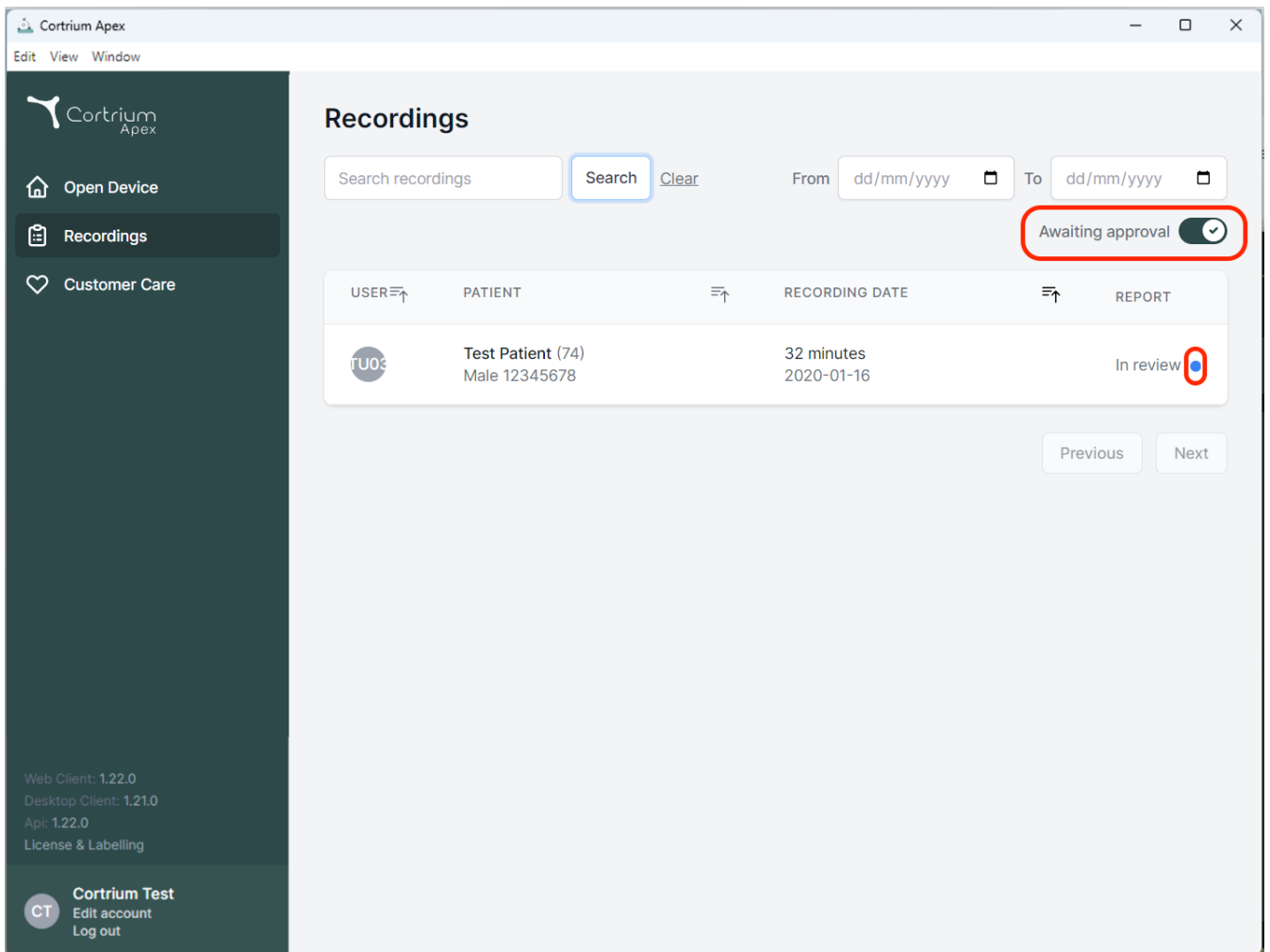**Recordings**

Search recordings   From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒

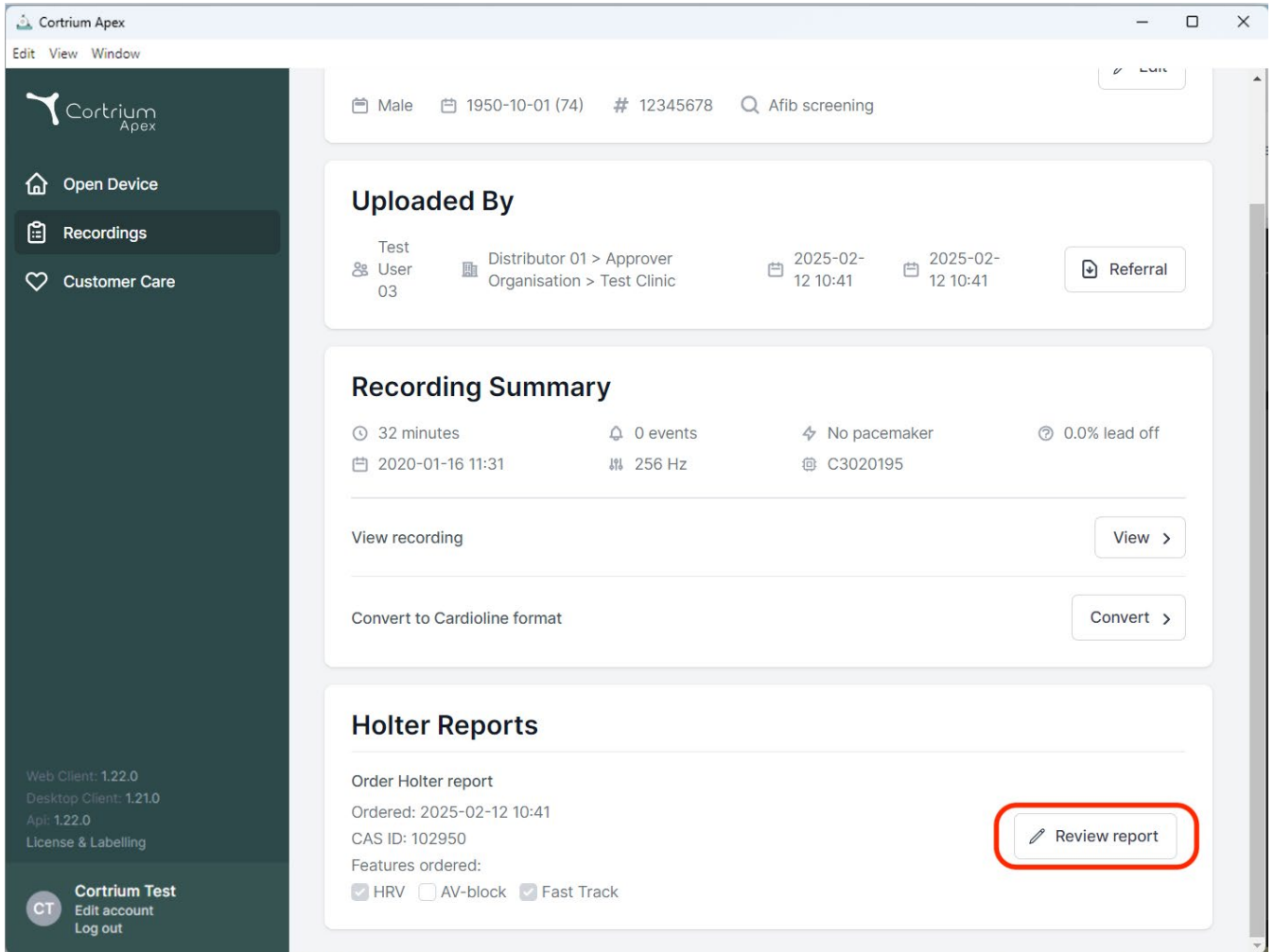
| USER | PATIENT                            | RECORDING DATE           | REPORT    |
|------|------------------------------------|--------------------------|-----------|
| TU03 | Test Patient (74)<br>Male 12345678 | 32 minutes<br>2020-01-16 | In review |

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
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area shows patient information at the top: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the "Uploaded By" section, showing the test user (User 03) and the distributor (Distributor 01 > Approver Organisation > Test Clinic), with upload dates of 2025-02-12 10:41 and a "Referral" button. The "Recording Summary" section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It includes buttons for "View recording" and "Convert to Cardioline format". The "Holter Reports" section shows the order details: "Order Holter report", "Ordered: 2025-02-12 10:41", "CAS ID: 102950", and "Features ordered: HRV, AV-block, Fast Track". A "Review report" button is highlighted with a red rectangle.

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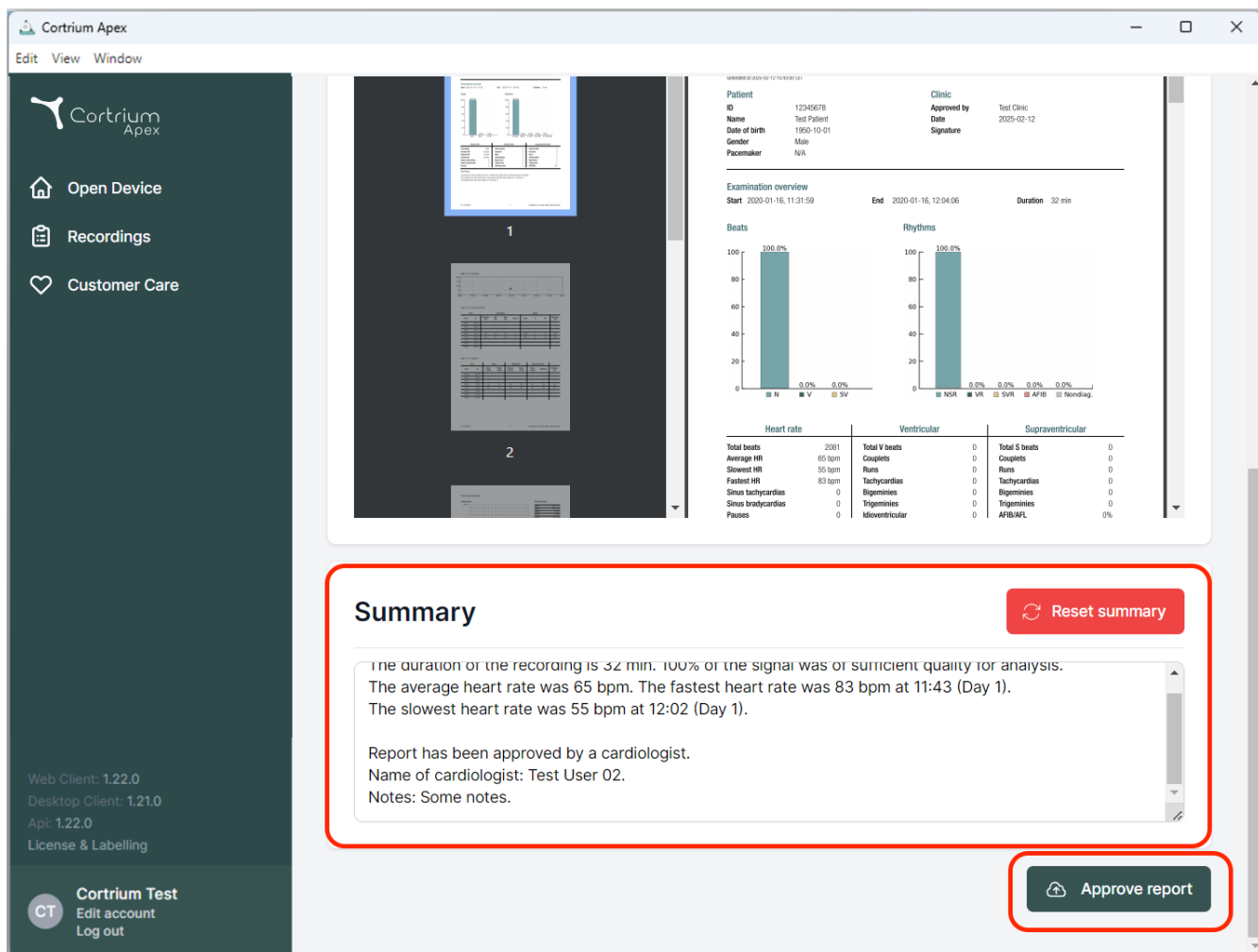The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: "Open Device", "Recordings", and "Customer Care". The main content area shows a patient report for "Test Patient" (ID: 12345678, DOB: 1950-10-01, Male, Pacemaker: N/A). The report includes an "Examination overview" with start/end times and duration, and two bar charts for "Beats" and "Rhythms" showing 100% completion. Below these are tables for "Heart rate", "Ventricular", and "Supraventricular" data. A red-bordered box highlights the "Summary" section, which contains the following text:

**Summary** Reset summary


The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.  
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).  
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.  
 Name of cardiologist: Test User 02.  
 Notes: Some notes.

At the bottom right, there is a button labeled "Approve report".

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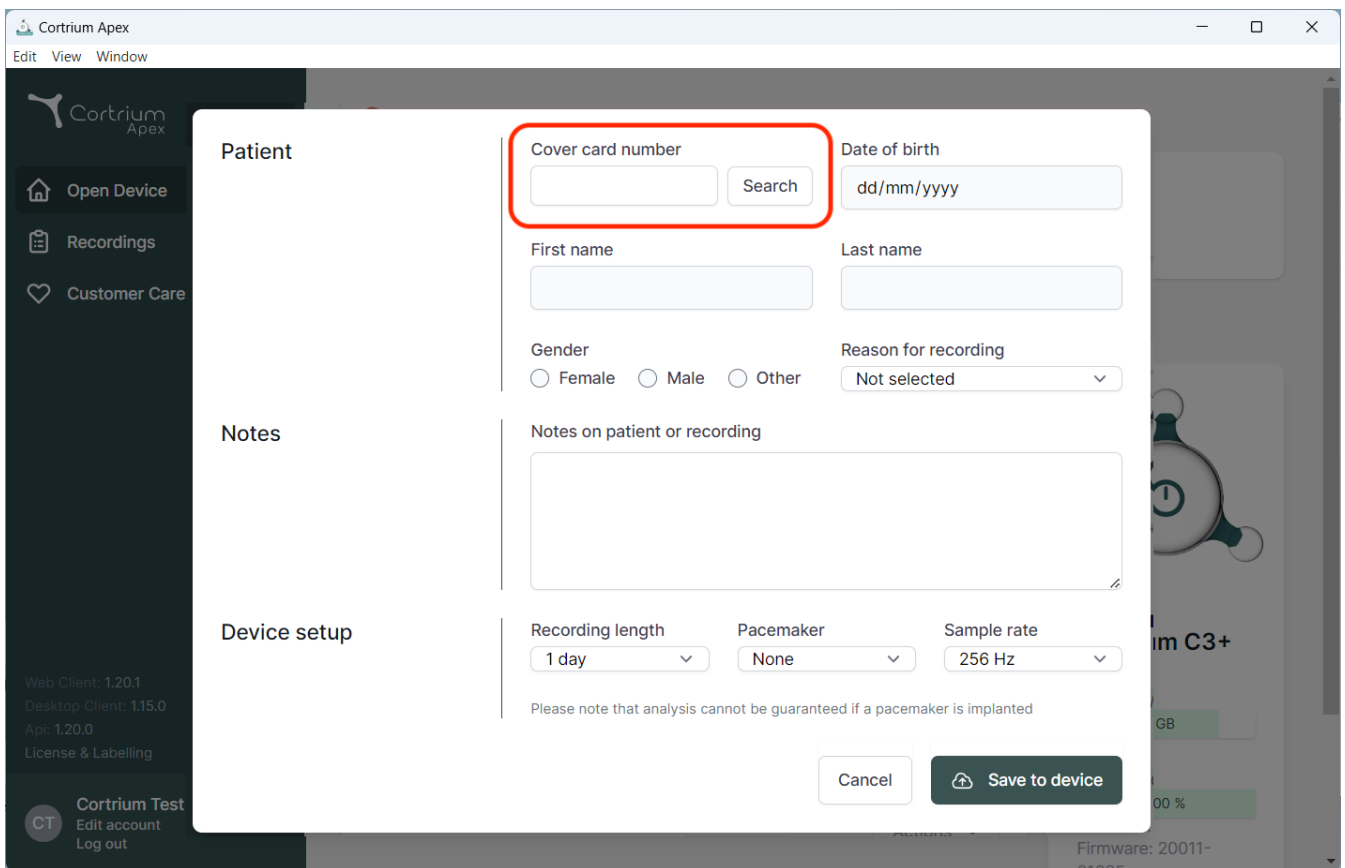The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying a form for patient information. The form is divided into sections: "Patient", "Notes", and "Device setup".

**Patient section:**

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

**Notes section:**

- Notes on patient or recording:** A large text area for notes.

**Device setup section:**

- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

Below the device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the form, there are two buttons: "Cancel" and "Save to device".

The background of the software shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". At the bottom left, there is a "Cortrium Test" section with links for "Edit account" and "Log out".

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