# Cortrium Apex 1.27

# Cortrium Apex 1.27

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
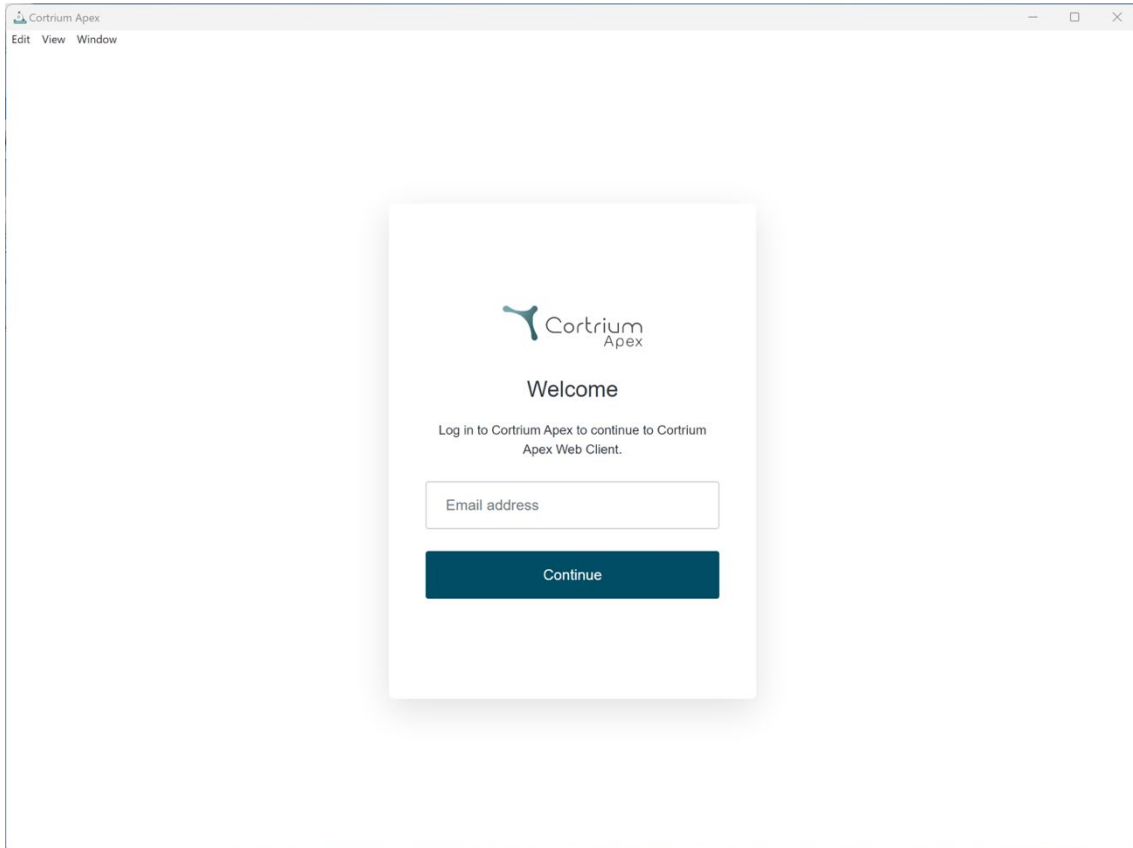
The screenshot shows a web browser window titled "Cortrium Apex". The browser's menu bar includes "Edit", "View", and "Window". The main content area displays a login form with the following elements:

- Cortrium Apex logo
- Section header: **Welcome**
- Text: "Log in to Cortrium Apex to continue to Cortrium Apex Web Client."
- Input field labeled "Email address"
- A dark blue button labeled "Continue"

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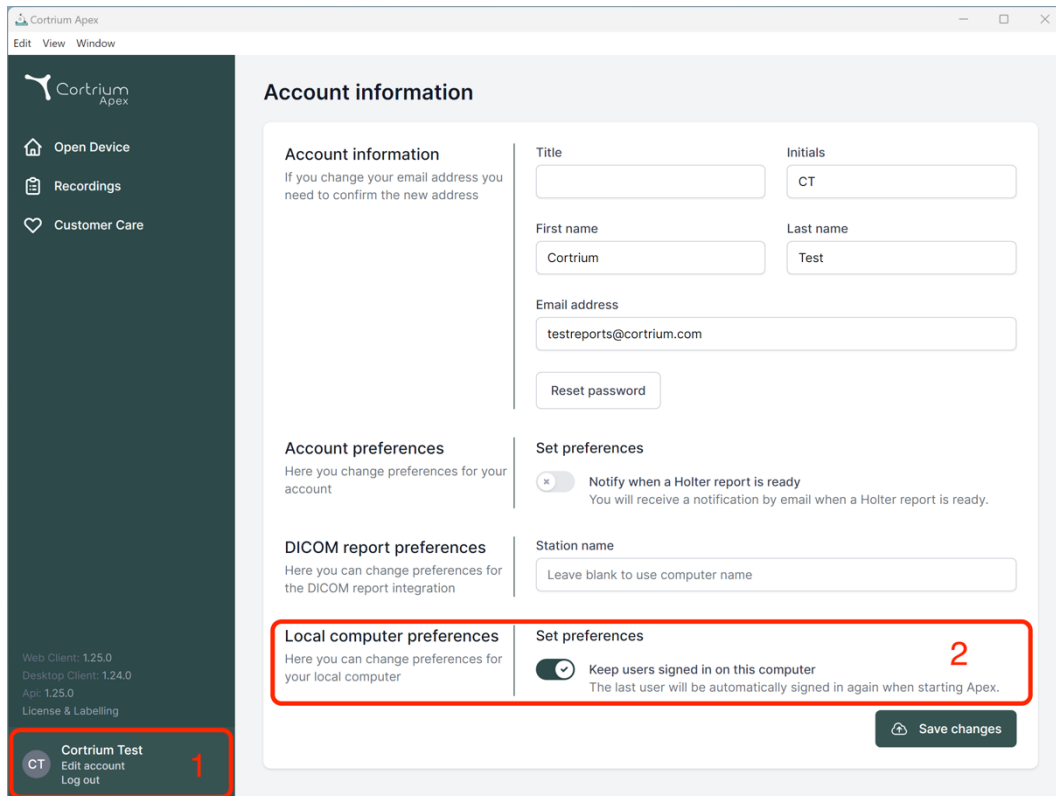The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client, Desktop Client, and API, along with a 'License & Labelling' link. At the bottom of the sidebar is a user profile card for 'Cortrium Test' with a red box labeled '1' around it, containing 'Edit account' and 'Log out' links.

The main content area is titled 'Account information'. It contains several sections:
 

- Account information:** Fields for Title, Initials (CT), First name (Cortrium), Last name (Test), and Email address (testreports@cortrium.com). A 'Reset password' button is also present.
- Account preferences:** A section for changing account preferences.
- DICOM report preferences:** A section for changing preferences for DICOM report integration.
- Local computer preferences:** A section for changing preferences for the local computer, highlighted with a red box labeled '2'. It includes a toggle switch for 'Keep users signed in on this computer' which is currently turned on. Below the toggle, it states: 'The last user will be automatically signed in again when starting Apex.'
- Set preferences:** A section for setting preferences, including a 'Station name' field with the placeholder 'Leave blank to use computer name'.


At the bottom right of the main content area is a 'Save changes' button.

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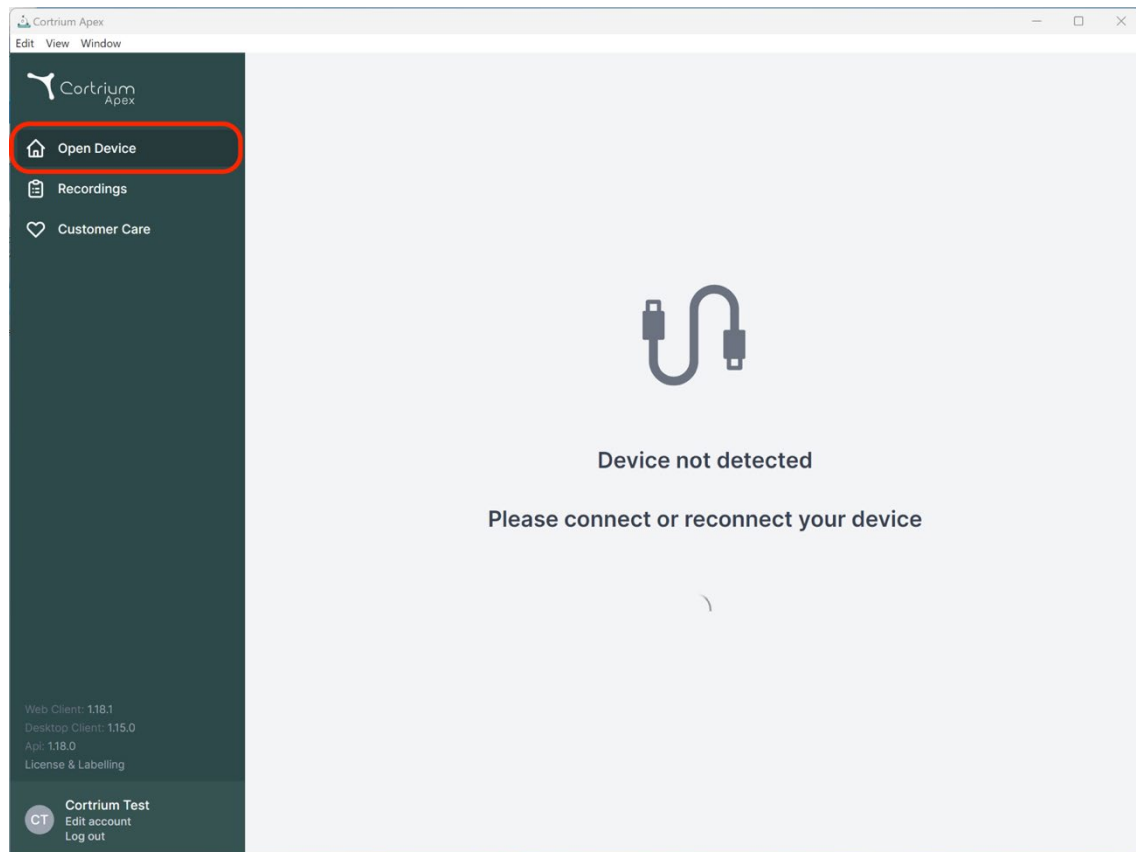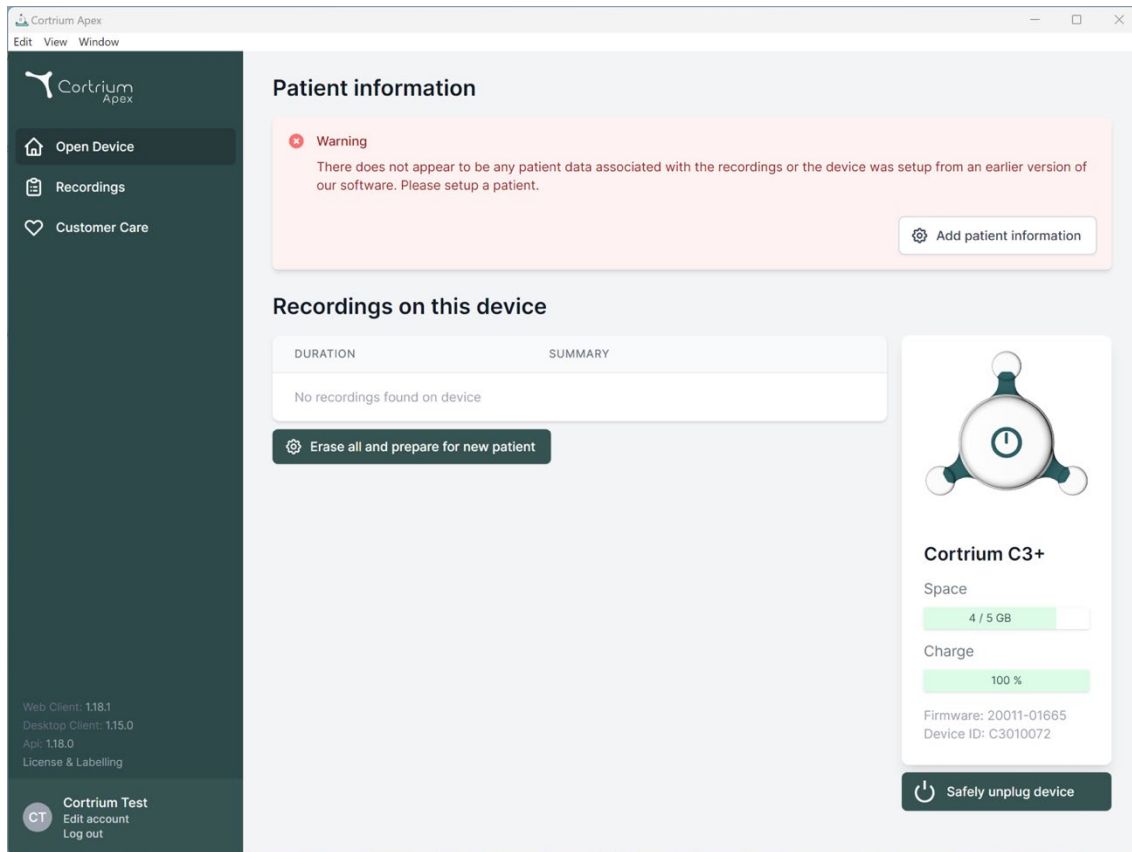
Cortrium Apex  
Edit View Window

**Patient information**

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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**Save to device**

**Warning** This will erase all recordings on the device

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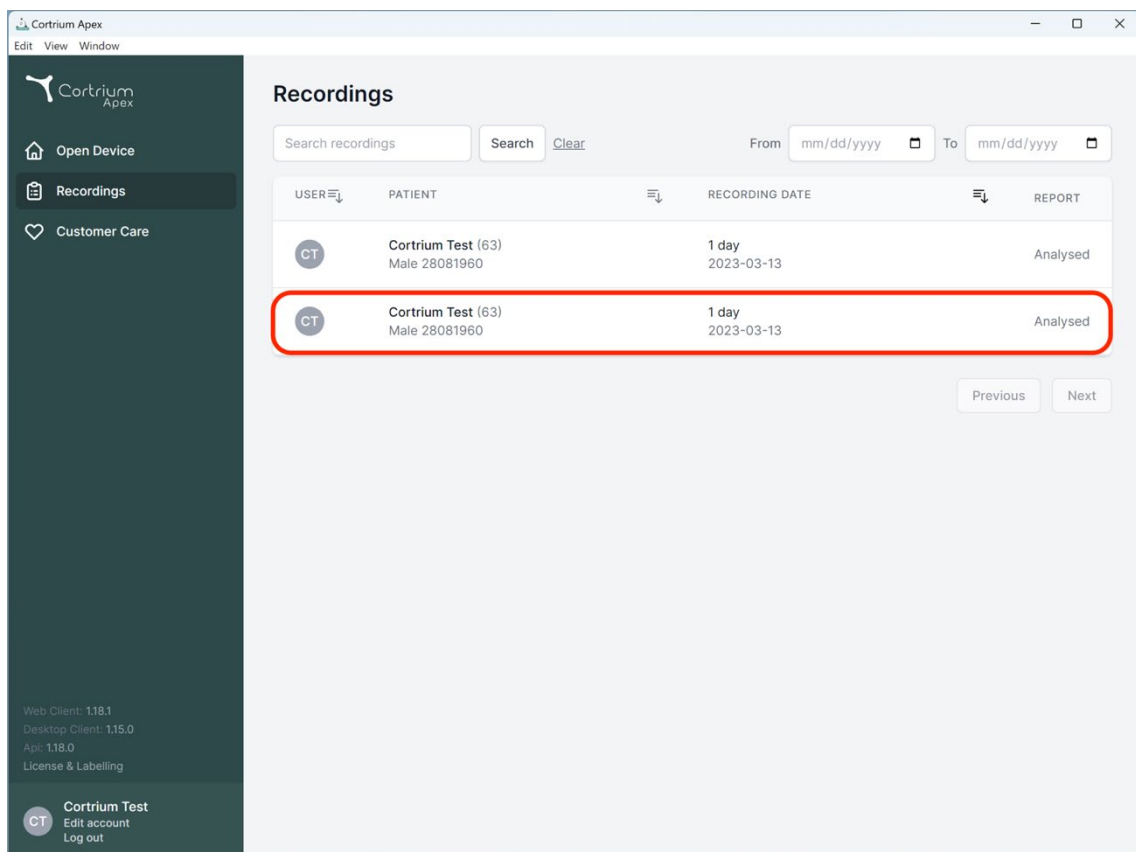
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area features a search bar and a table of recordings. The table has columns for 'USER', 'PATIENT', 'RECORDING DATE', and 'REPORT'. Two identical entries are listed, both for 'Cortrium Test (63)' with a recording date of '2023-03-13' and a status of 'Analysed'. The second entry is highlighted with a red border. At the bottom of the table, there are 'Previous' and 'Next' navigation buttons. The sidebar also shows version information: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', and 'App: 1.18.0', along with a 'License & Labelling' link. The user is logged in as 'Cortrium Test' and can 'Edit account' or 'Log out'.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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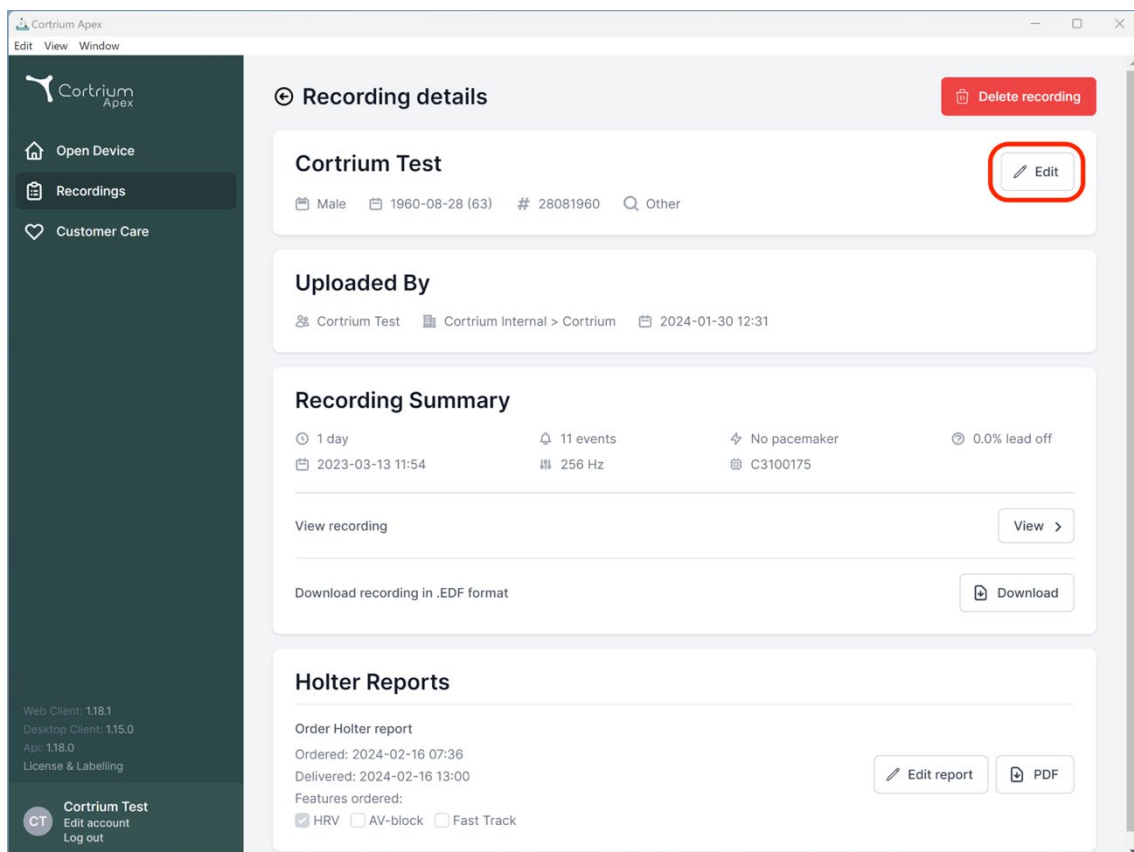
2023-03-13 11:54 | 256 Hz | C3100175

View recording View >

Download recording in .EDF format Download

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
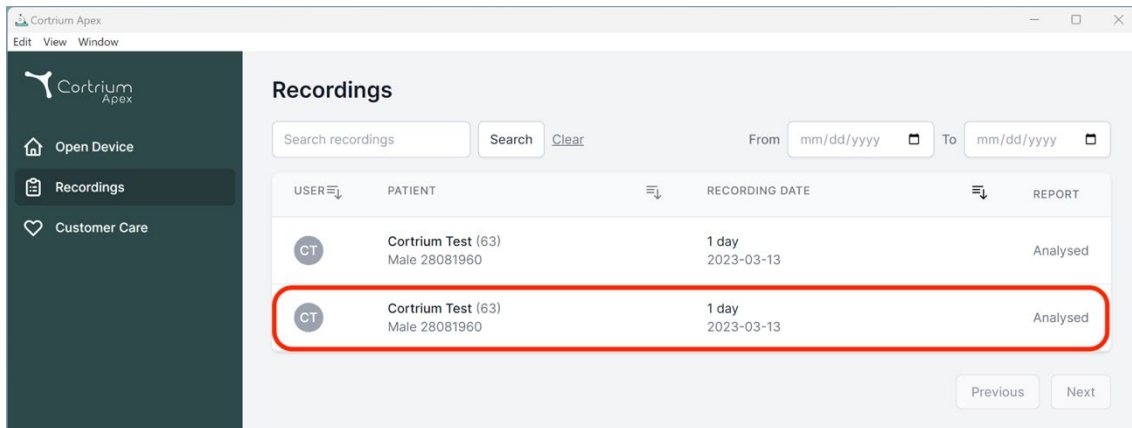
**Recordings**

Search recordings  Search [Clear](#) From  To

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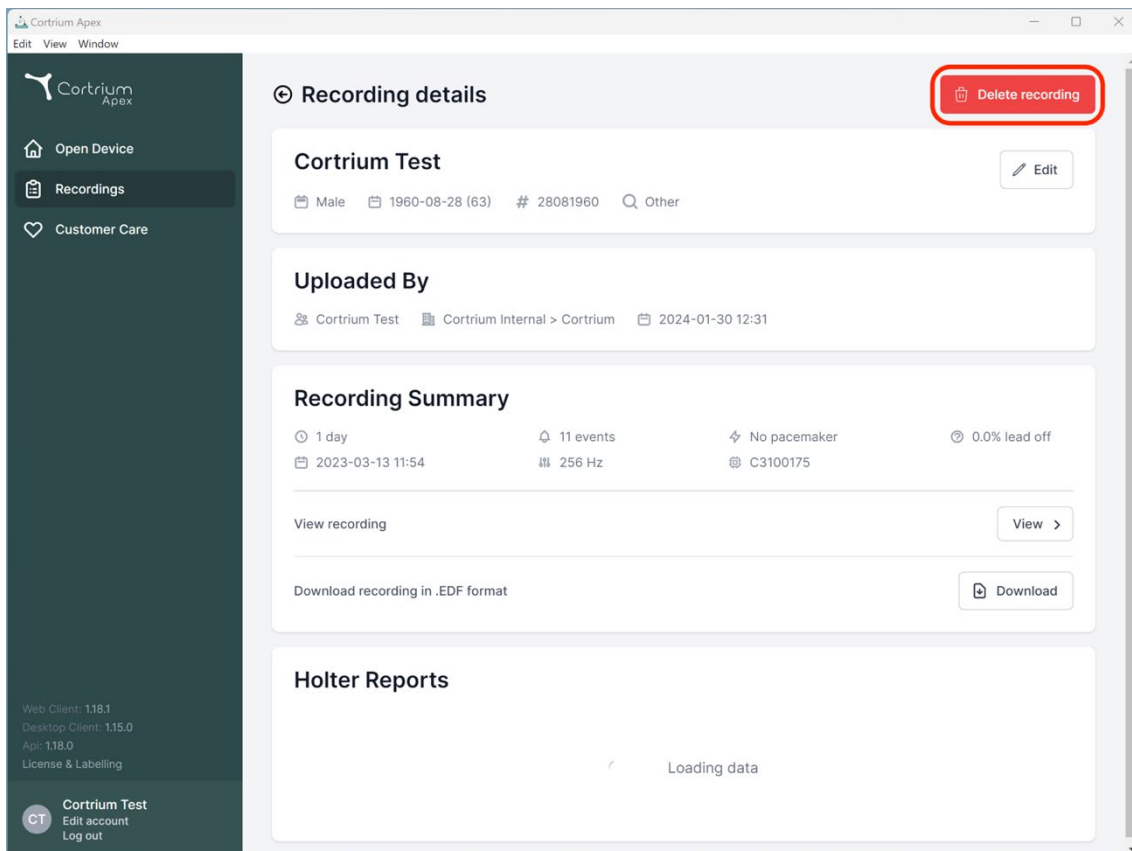Previous Next

**Recording details** [Delete recording](#)

**Cortrium Test** [Edit](#)

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

View recording [View](#)

Download recording in .EDF format [Download](#)

**Holter Reports**

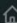
Loading data

Web Client: 1.18.1  
Desktop Client: 1.15.0  
Api: 1.18.0  
License & Labelling

CT Cortrium Test  
Edit account  
Log out

Cortrium Apex  
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

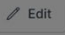
 Customer Care

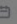
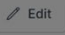
Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

 Cortrium Test  
Edit account  
Log out

### Recording details

**Cortrium Test** 

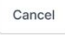
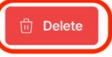
 Male  1960-08-28 (63)  # 28081960  Other


### Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

**Confirm delete**  
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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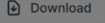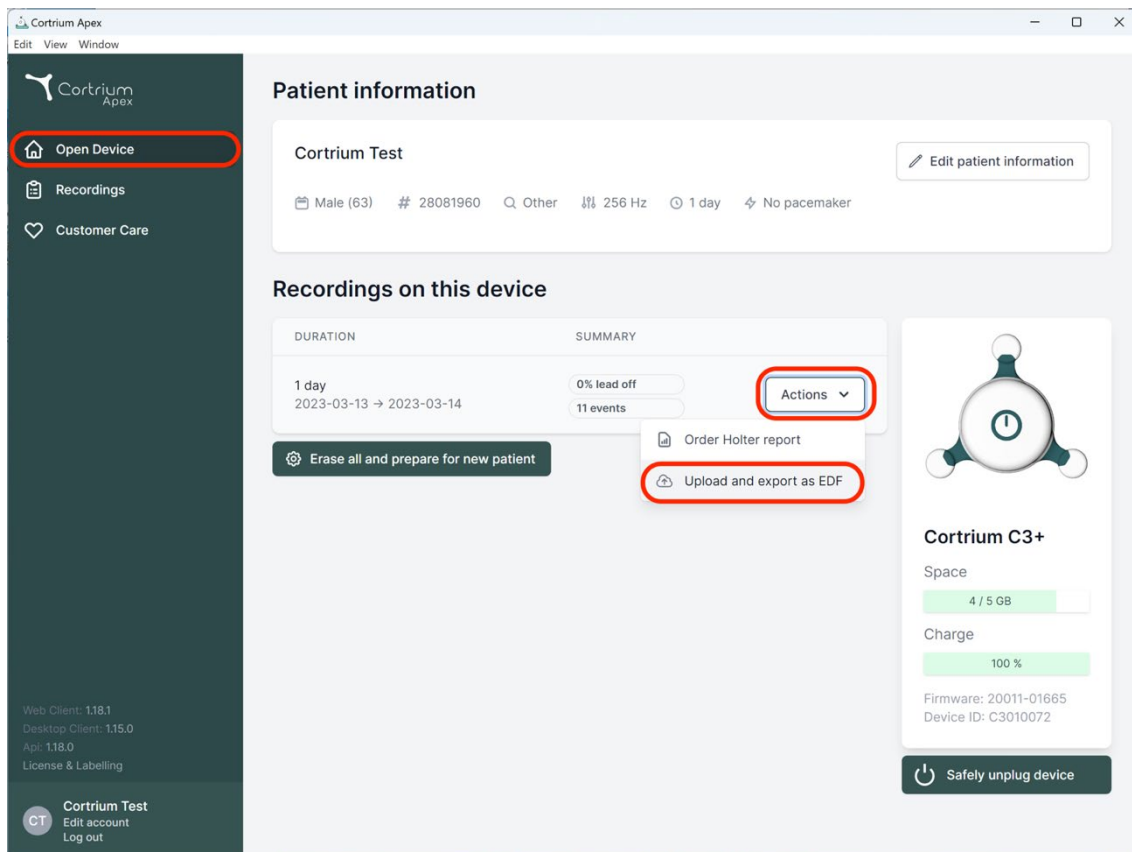
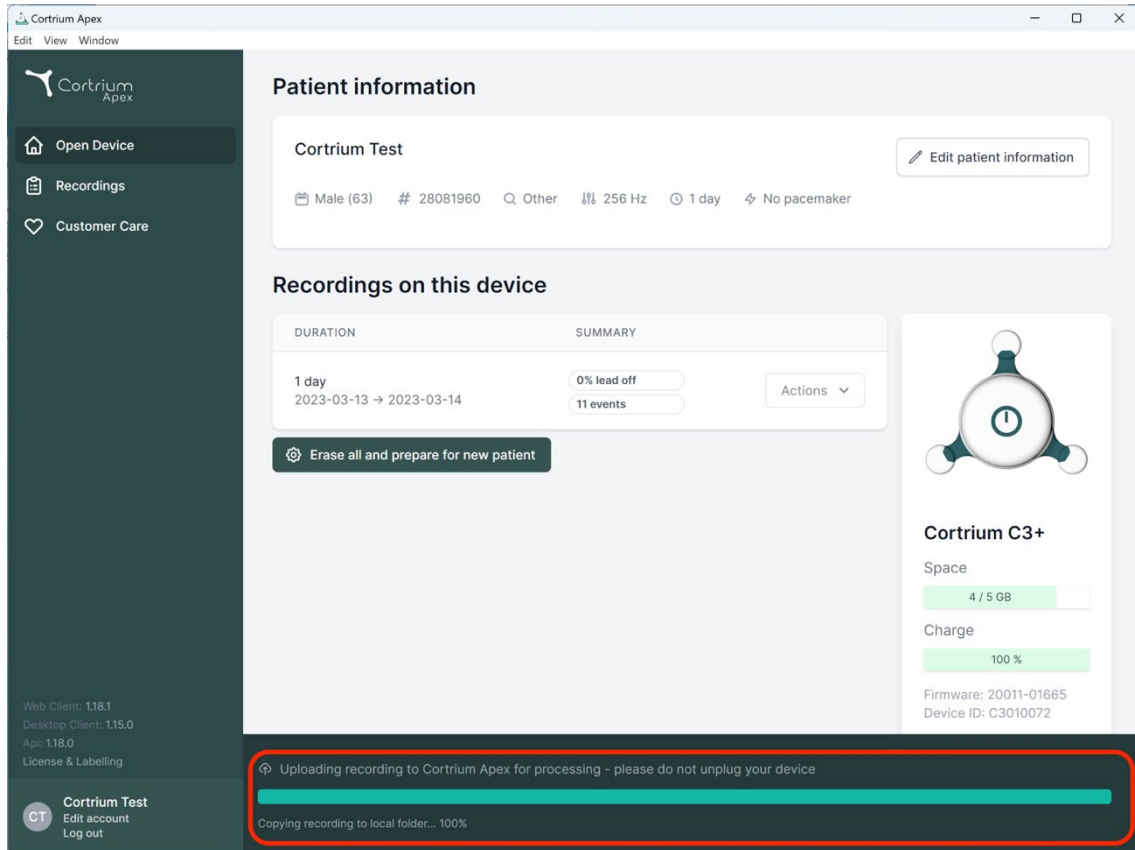
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">Actions</a>

[Erase all and prepare for new patient](#)

**Cortrium C3+**

Space  
4 / 5 GB

Charge  
100 %

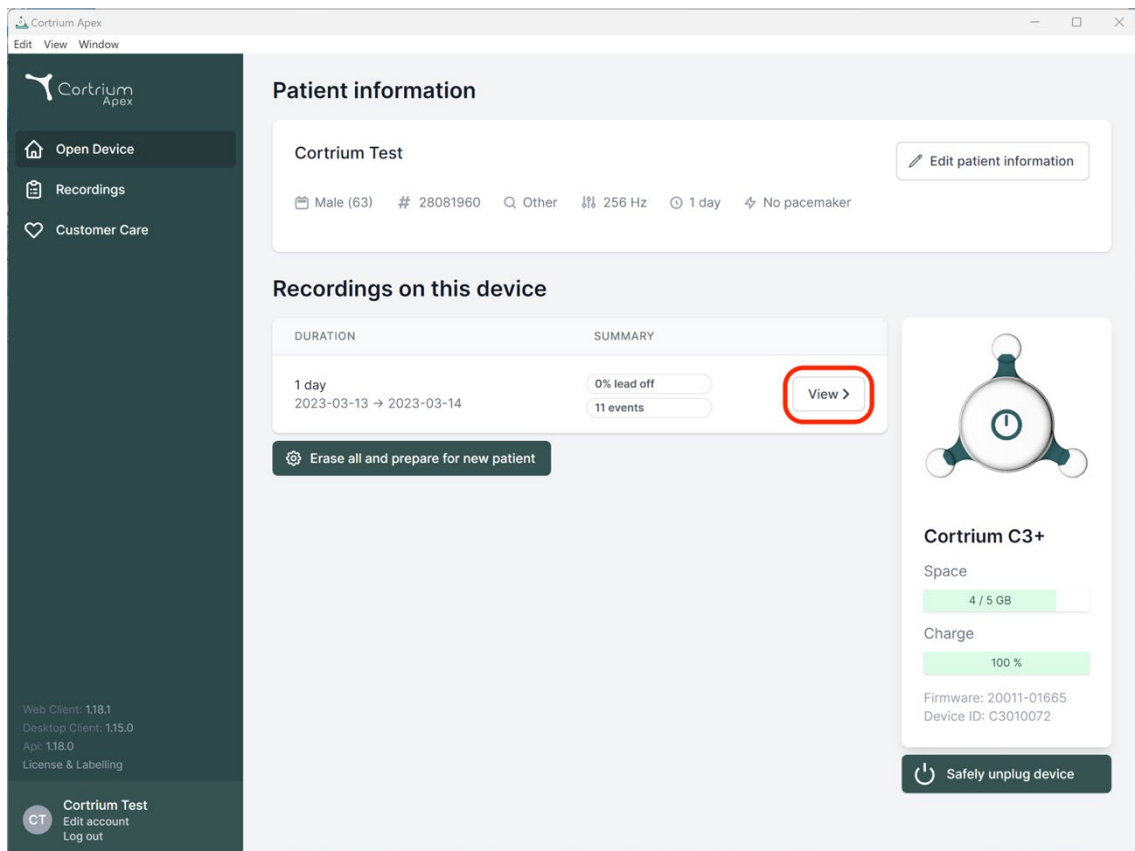
Firmware: 20011-01665  
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

**Cortrium Test**  
Edit account  
Log out

**Cortrium Apex**  
Edit View Window

**Patient information**

**Cortrium Test** [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">View &gt;</a>

[Erase all and prepare for new patient](#)

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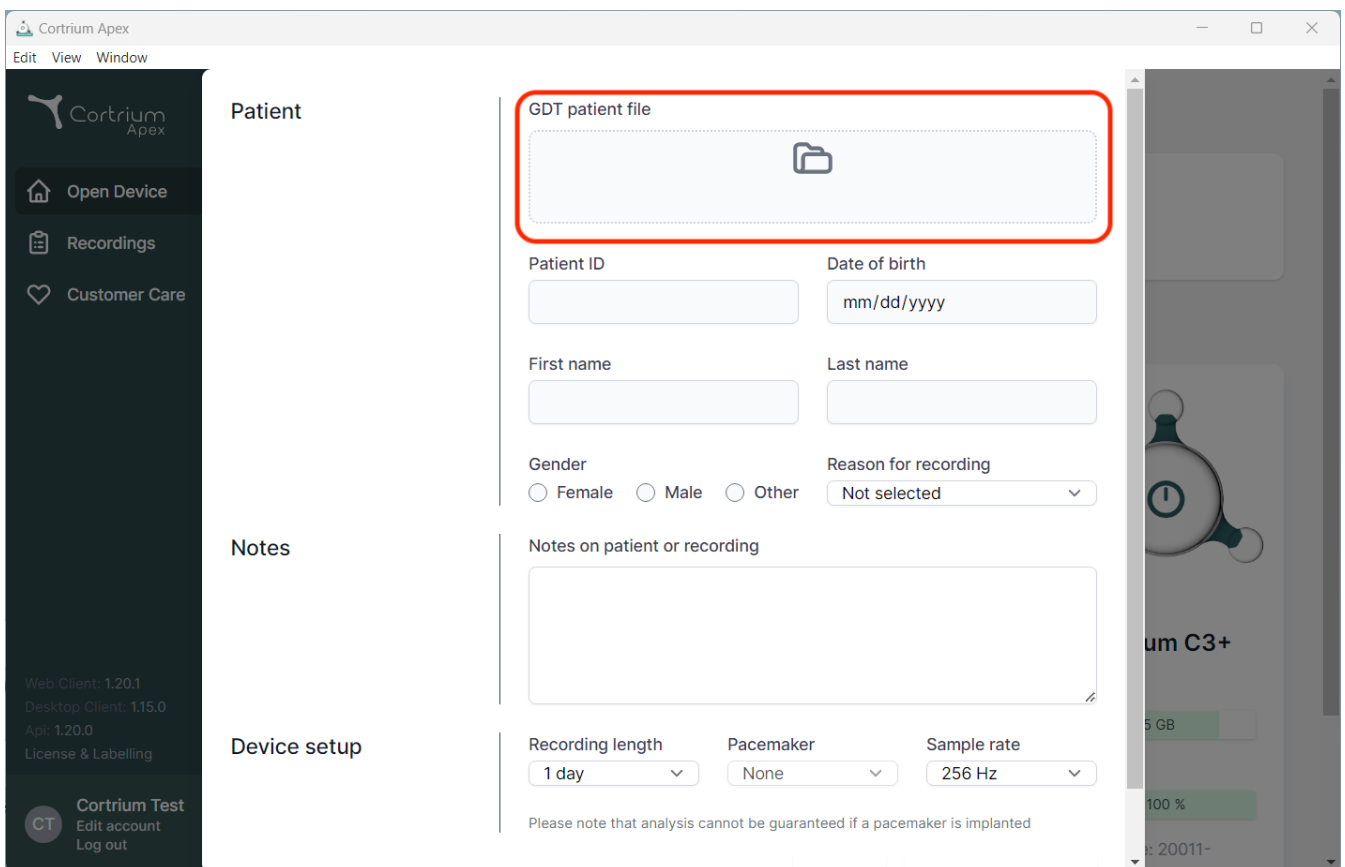
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section is highlighted and contains a form for entering patient information. The form includes a red-bordered box for uploading a GDT patient file, followed by input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, and Last name. Below these are radio buttons for Gender (Female, Male, Other) and a dropdown menu for Reason for recording (Not selected). The Notes section has a text area for notes on the patient or recording. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

Cortrium Apex

Edit View Window

Cortrium Apex

Patient

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None

Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
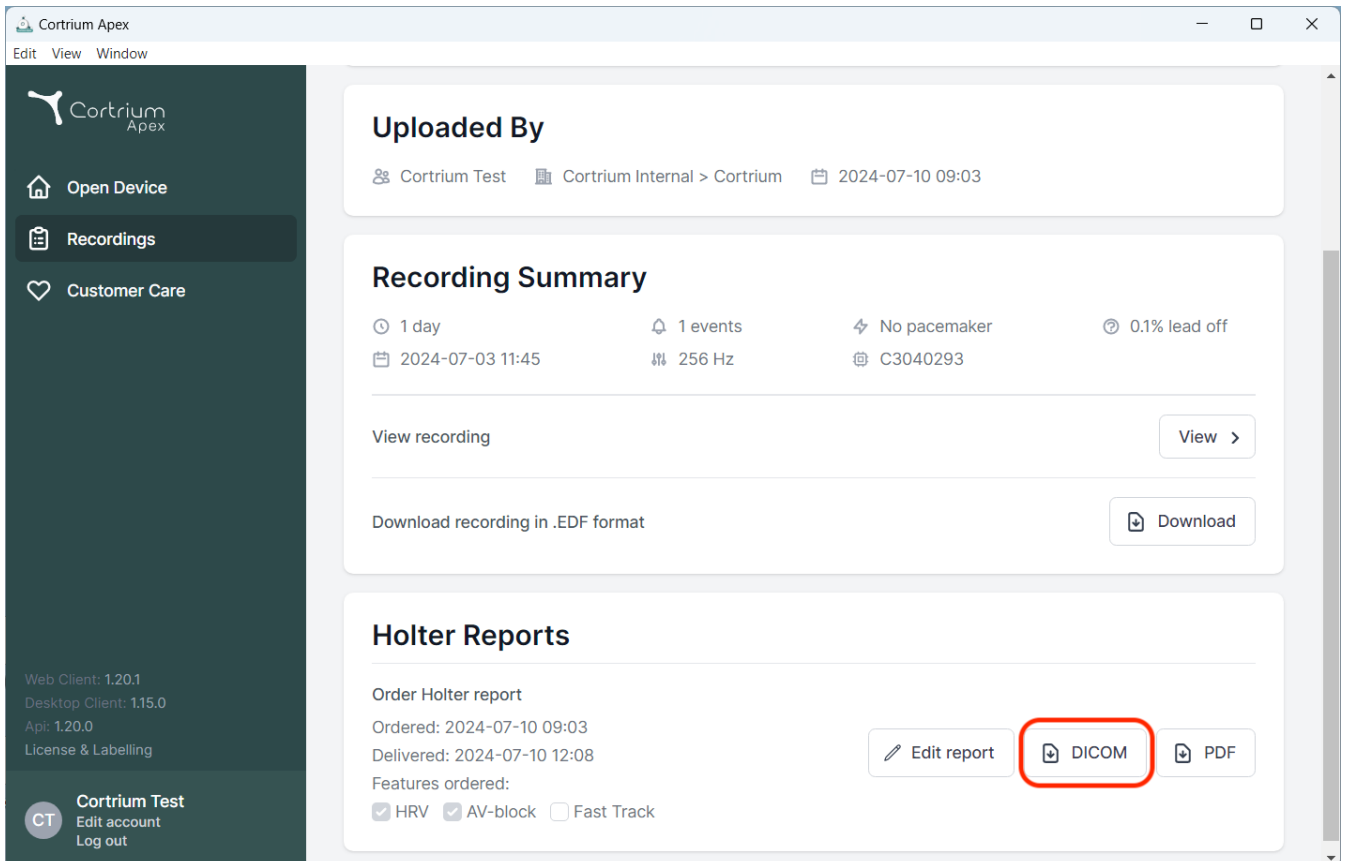
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0, License & Labelling) and a user profile for 'Cortrium Test'. The main content area has a top section 'Uploaded By' showing 'Cortrium Test' and a date. Below is the 'Recording Summary' section with various parameters like '1 day', '1 events', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'. The bottom section is 'Holter Reports', showing 'Order Holter report' details and a row of buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

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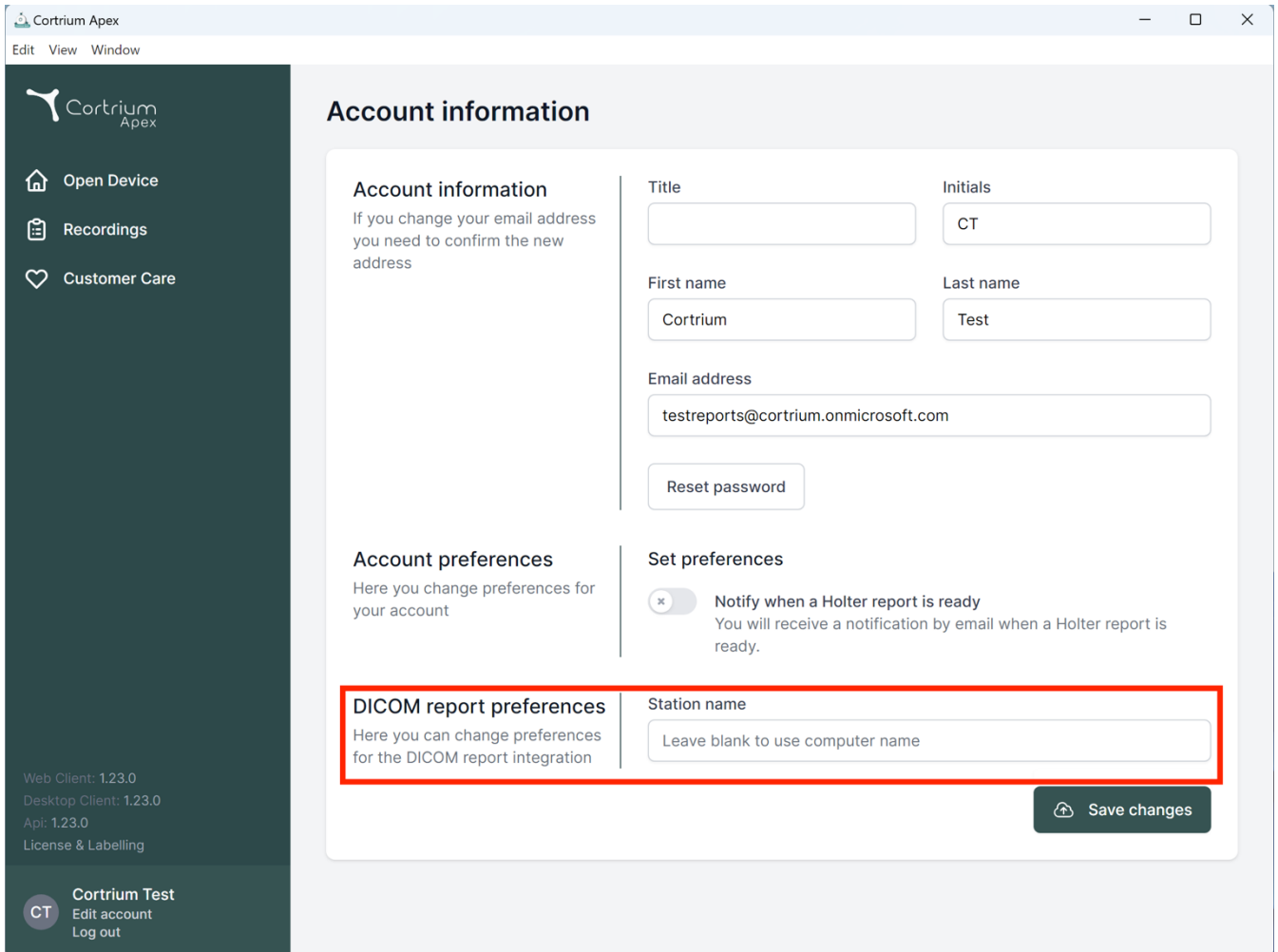## Account information

### Account information

If you change your email address you need to confirm the new address

Title

Initials

CT

First name

Cortrium

Last name

Test

Email address

testreports@cortrium.onmicrosoft.com

Reset password

### Account preferences

Here you change preferences for your account

### Set preferences

×

Notify when a Holter report is ready

You will receive a notification by email when a Holter report is ready.

### DICOM report preferences

Here you can change preferences for the DICOM report integration

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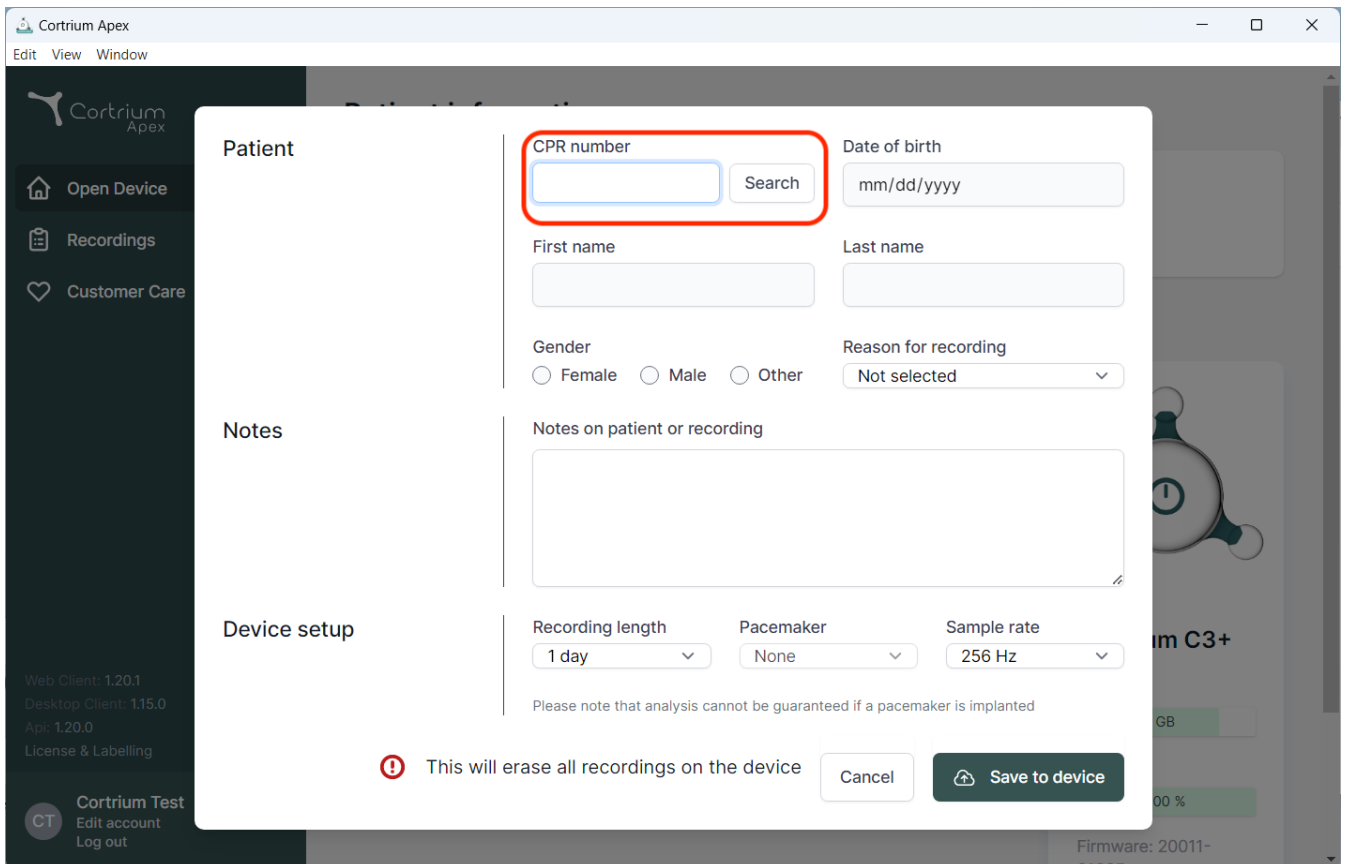
Pacemaker

Sample rate

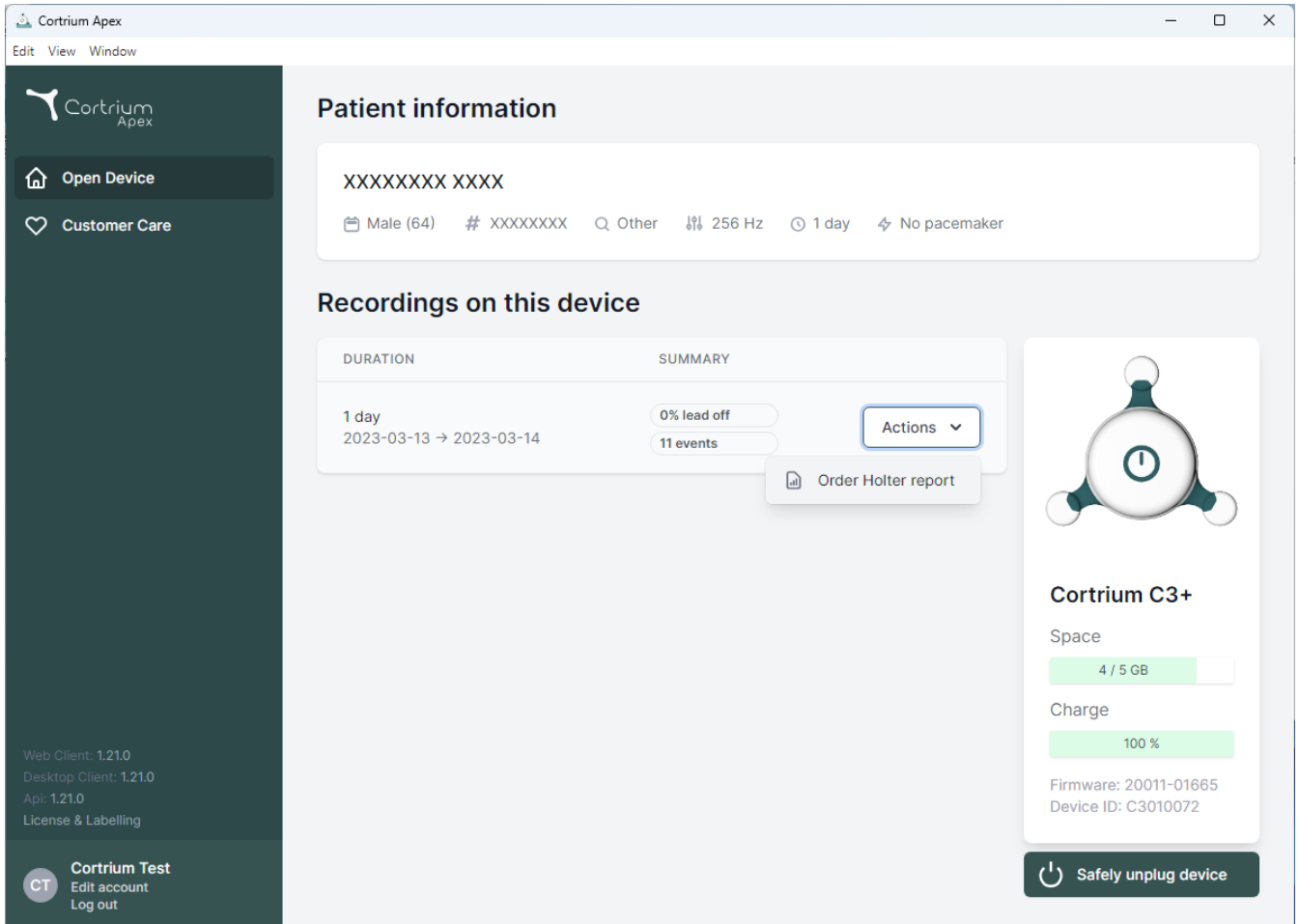
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: 'Open Device' and 'Customer Care'. The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

**Patient information**

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

**Cortrium C3+**

Space  
4 / 5 GB

Charge  
100 %

Firmware: 20011-01665  
Device ID: C3010072

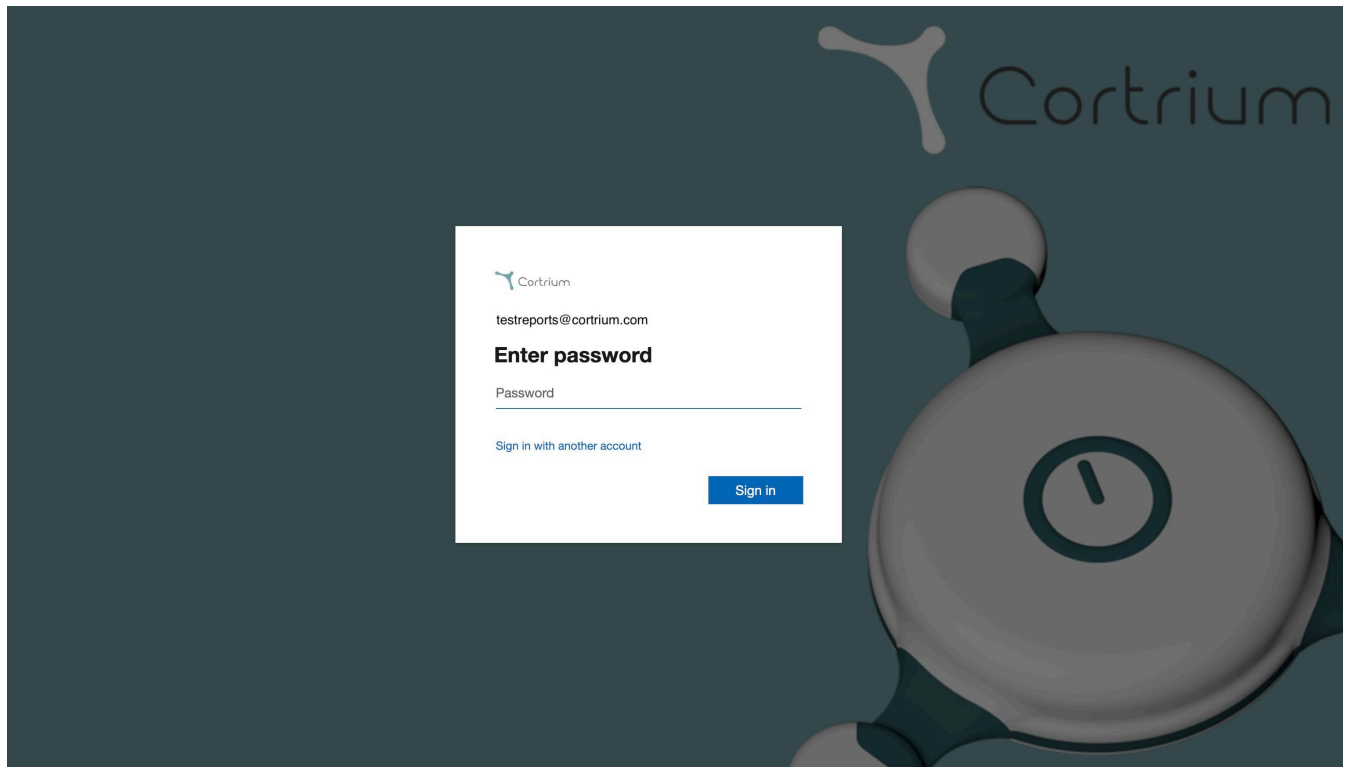
Safely unplug device

Web Client: 1.21.0  
Desktop Client: 1.21.0  
Api: 1.21.0  
License & Labelling

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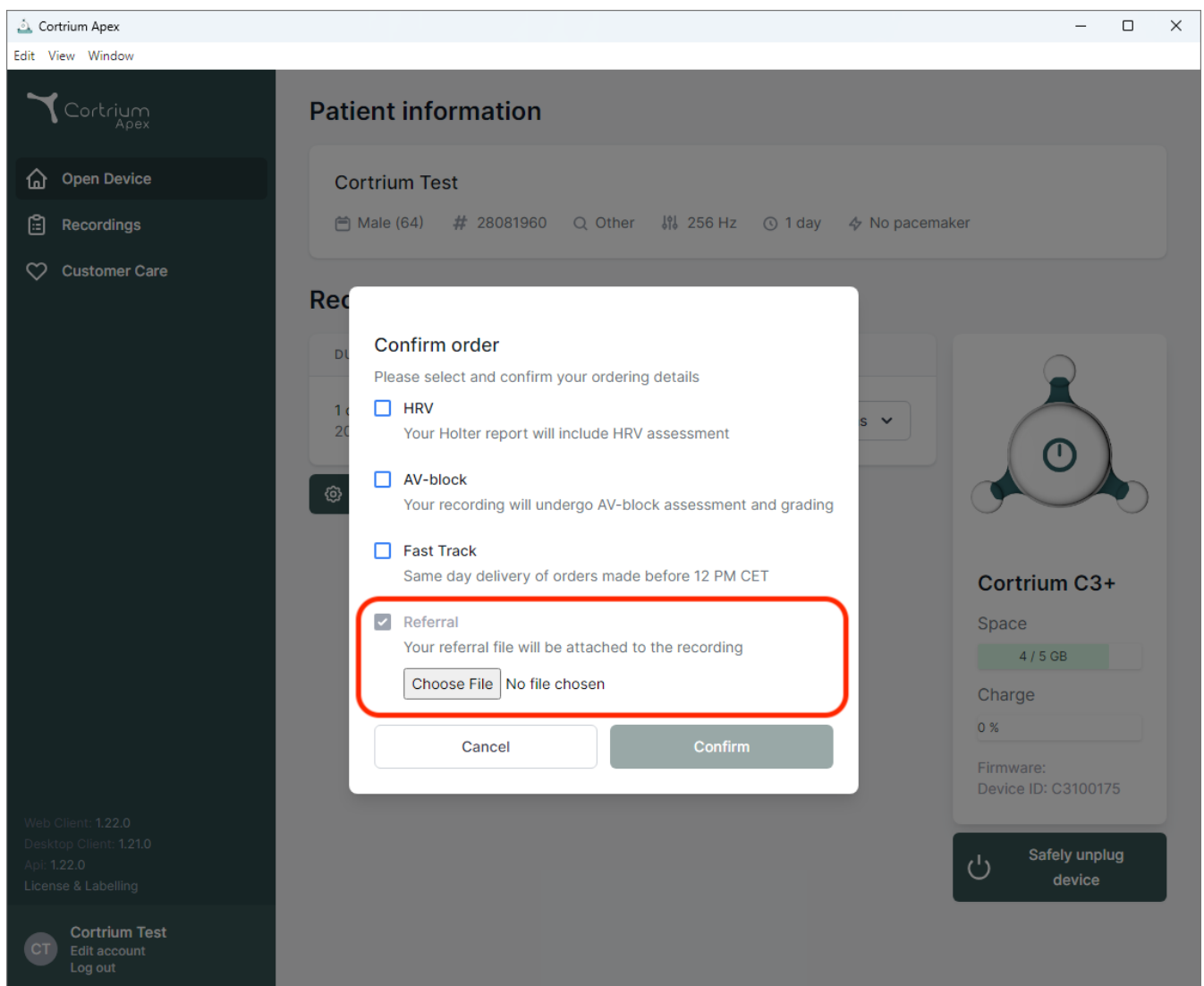
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
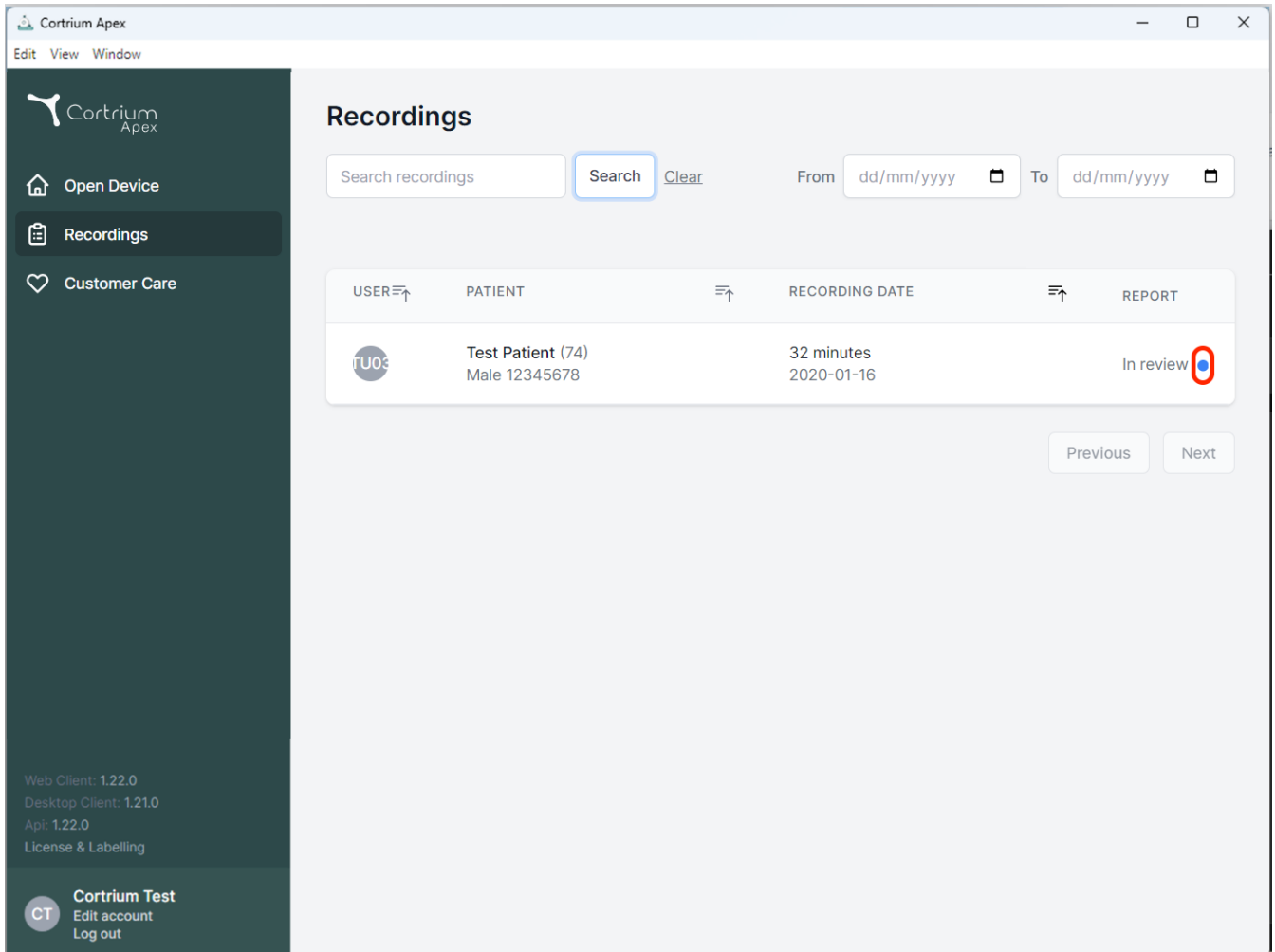
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area is titled "Recordings" and features a search bar with a "Search" button (highlighted with a blue box) and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a status of "In review". A blue dot is visible next to the "In review" status. At the bottom right of the table are "Previous" and "Next" buttons. The bottom left of the sidebar shows version information and a "Cortrium Test" section with "Edit account" and "Log out" links.

USER	PATIENT	RECORDING DATE	REPORT
ITU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review <span style="color: blue;">●</span>

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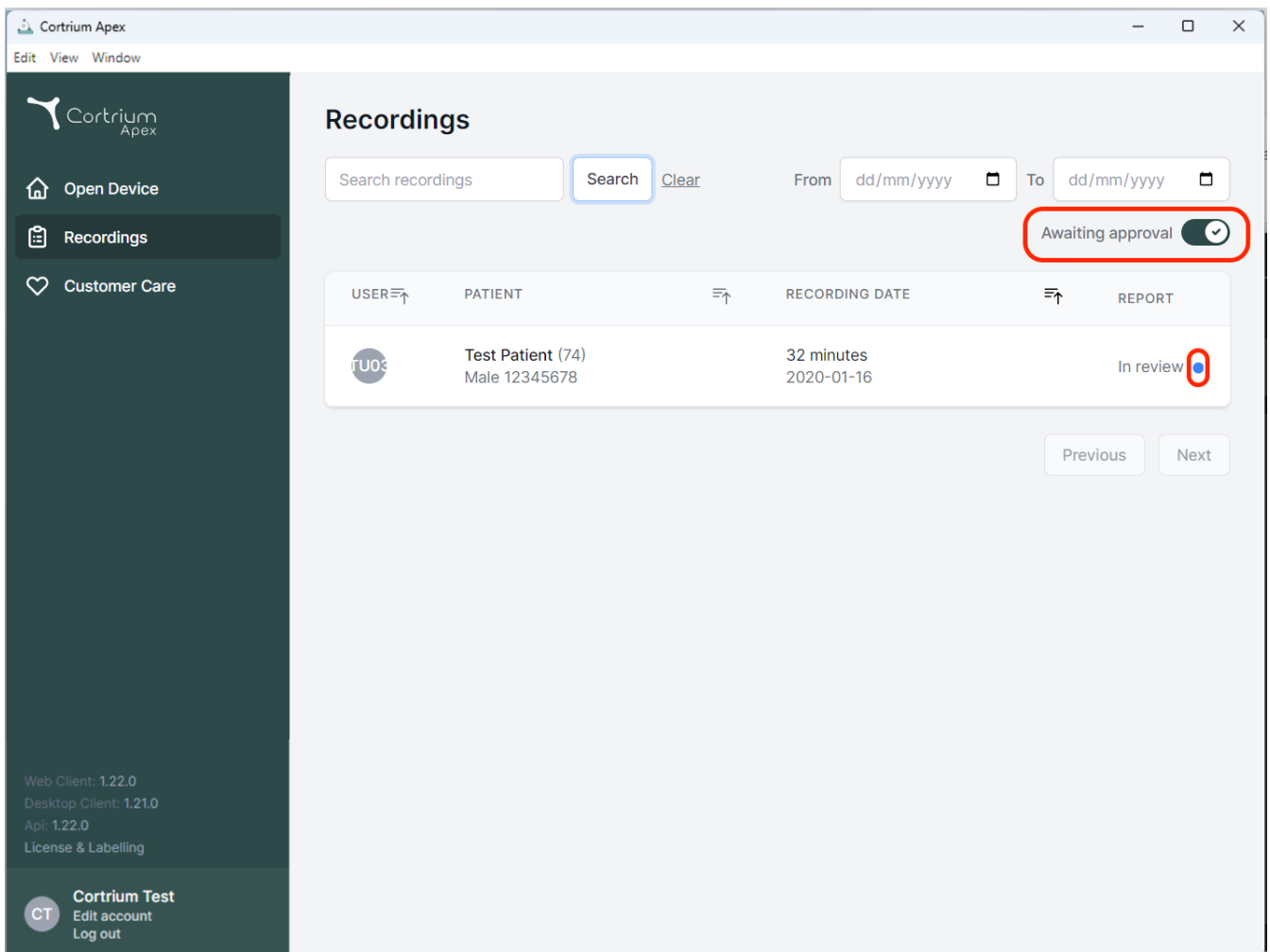Search recordings   From  To

Awaiting approval ☒

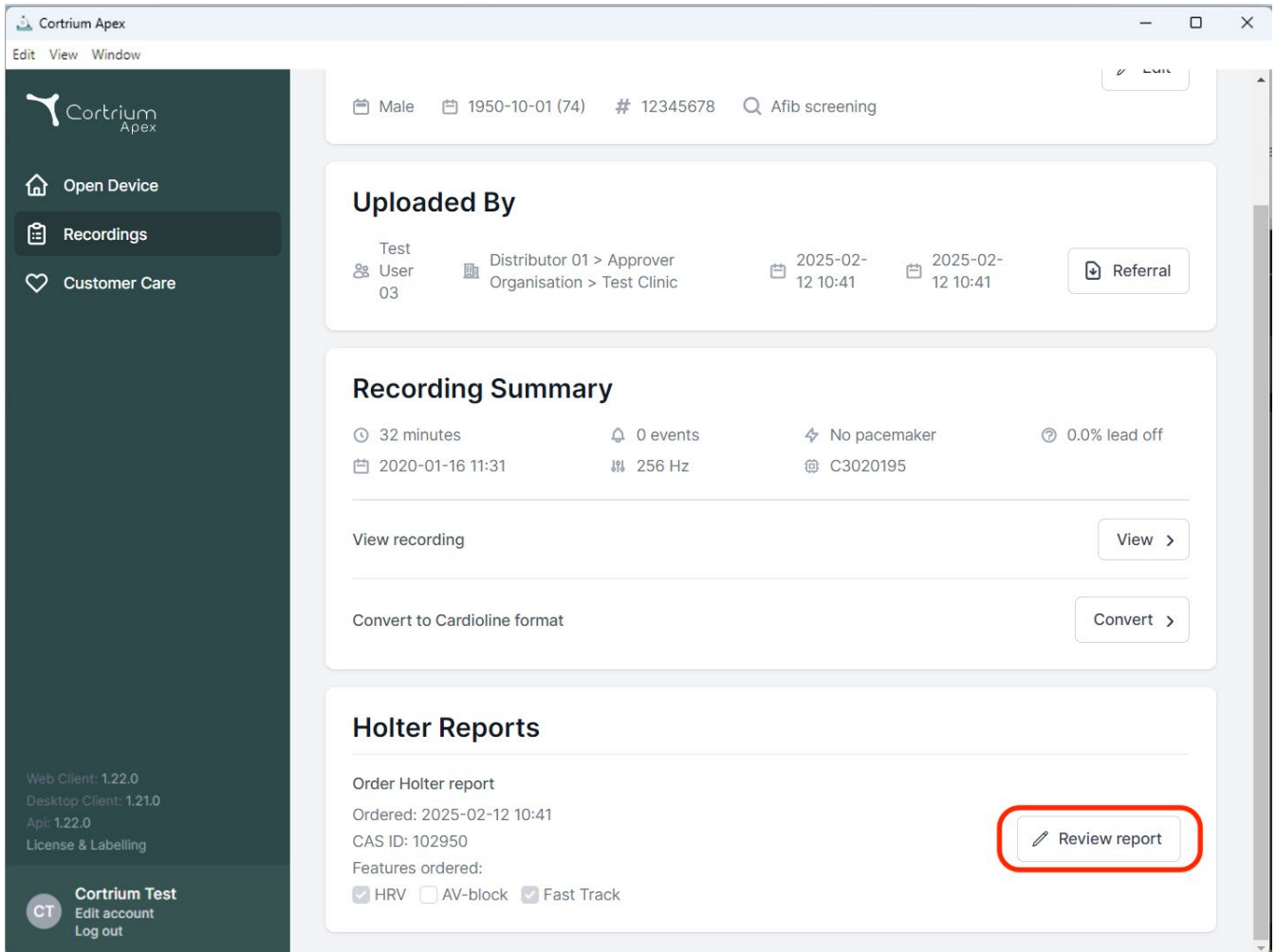
USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review <span style="color: blue;">●</span>

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with timestamps '2025-02-12 10:41' and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

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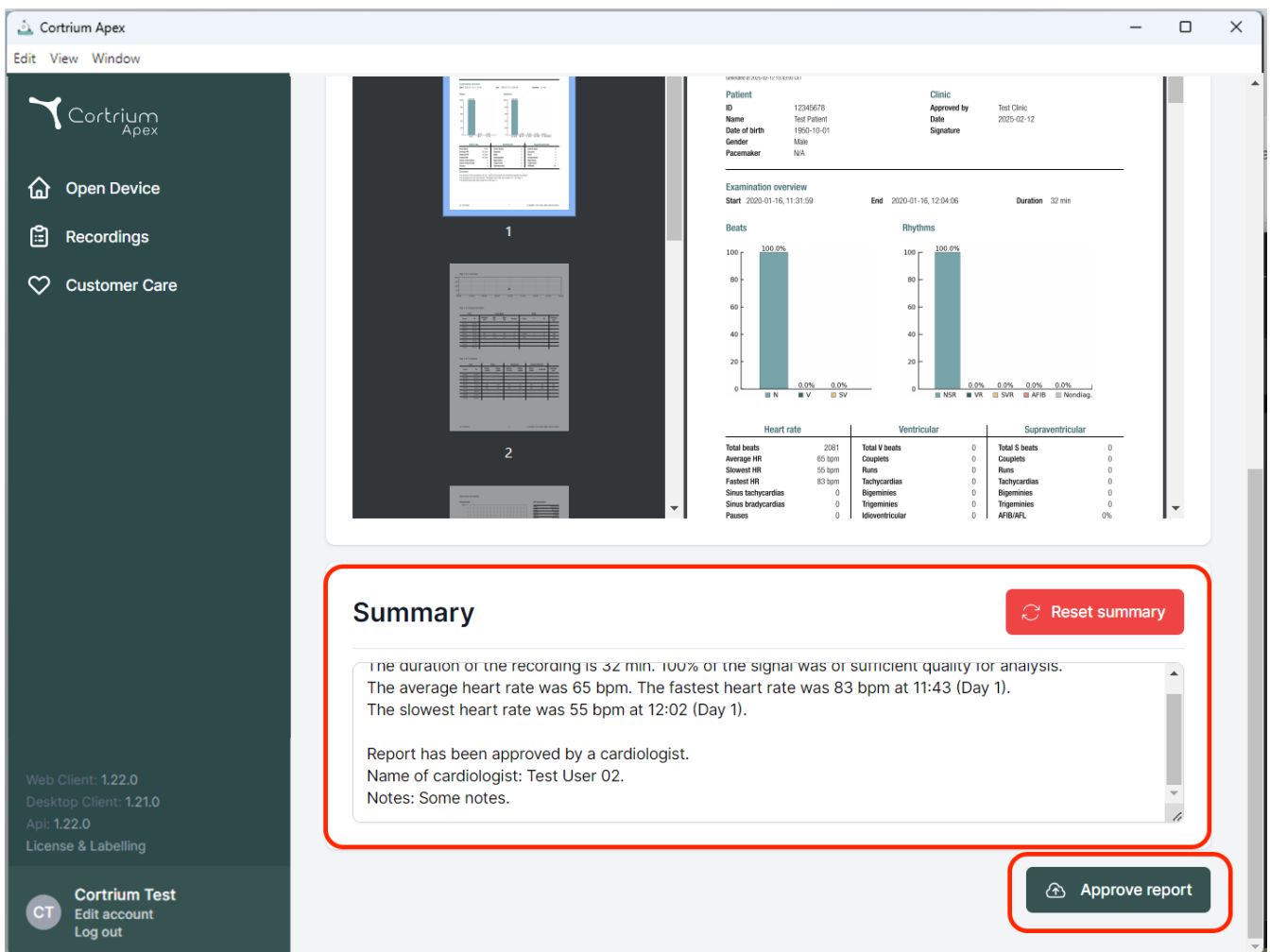**Cortrium Apex**  
Edit View Window

**Open Device**  
**Recordings**  
**Customer Care**

Web Client: 1.22.0  
Desktop Client: 1.21.0  
Api: 1.22.0  
License & Labelling

**Cortrium Test**  
Edit account  
Log out

**Summary** Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.  
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).  
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.  
Name of cardiologist: Test User 02.  
Notes: Some notes.

Approve report

**Patient Information:**  
ID: 12345678  
Name: Test Patient  
Date of birth: 1995-10-01  
Gender: Male  
Pacemaker: N/A  
Clinic: Test Clinic  
Approved by: Test Clinic  
Date: 2025-02-12  
Signature:

**Examination overview**  
Start: 2020-01-16, 11:31:59  
End: 2020-01-16, 12:04:06  
Duration: 32 min

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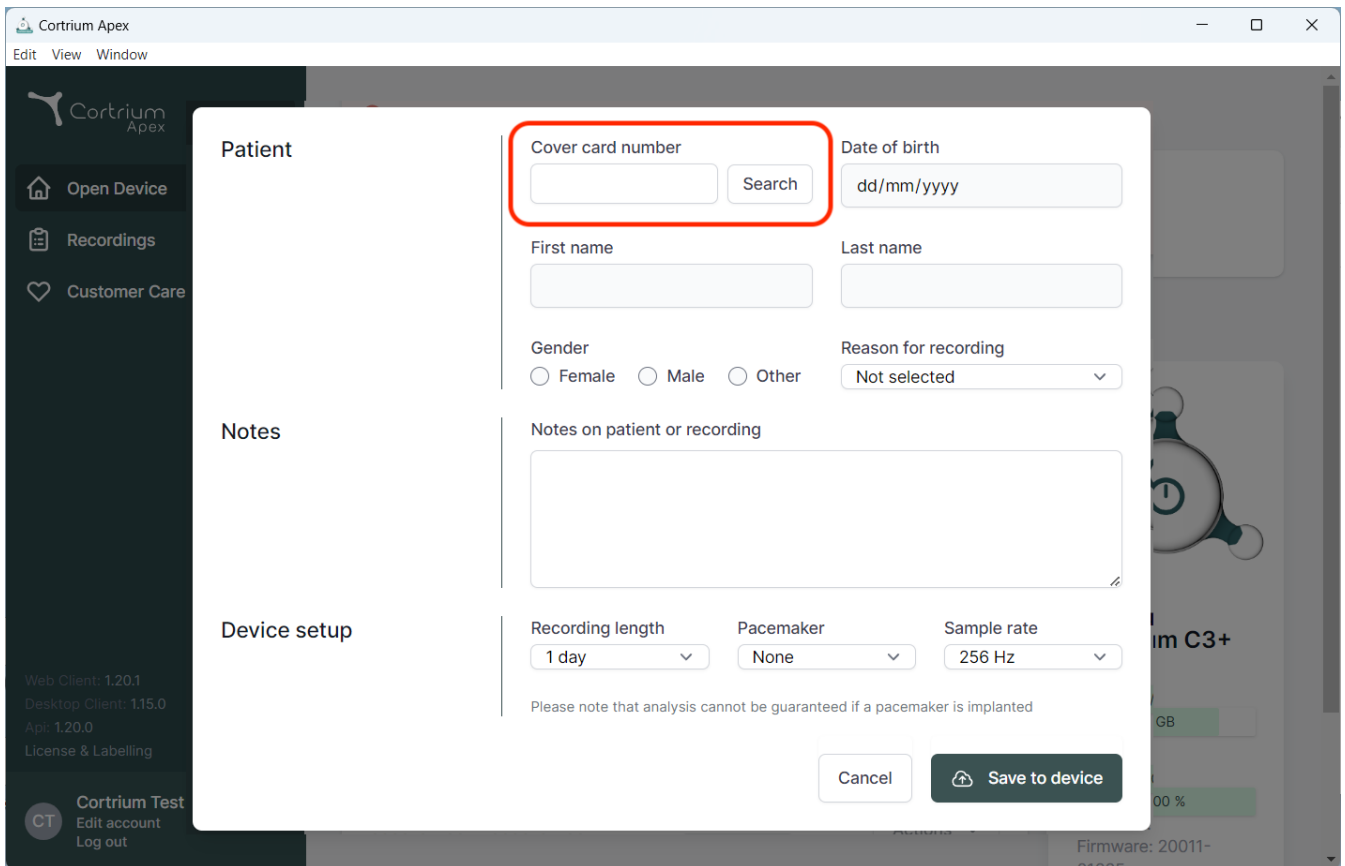
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for patient search and setup. The form is divided into sections: "Patient", "Notes", and "Device setup".

**Patient Section:**

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red box.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

**Notes Section:**

- Notes on patient or recording:** A large text area for entering notes.

**Device setup Section:**

- Recording length:** A dropdown menu currently showing "1 day".
- Pacemaker:** A dropdown menu currently showing "None".
- Sample rate:** A dropdown menu currently showing "256 Hz".

Below the device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

The background of the application shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar contains version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right corner of the sidebar shows a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark  
E-Mail: [info@cortrium.com](mailto:info@cortrium.com) • Website: [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033

**CE**  
**0123**  
47 / 47