Cortrium Apex 1.27

Cortrium Apex 1.27

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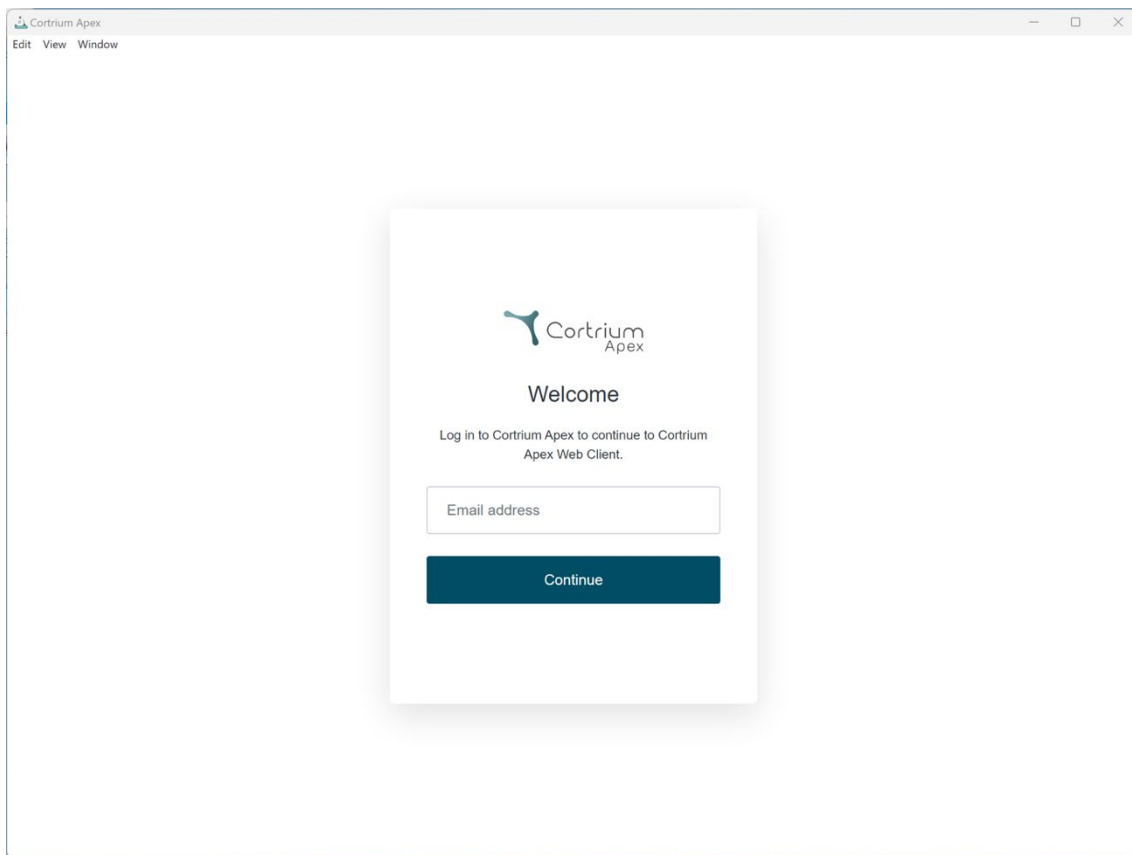
 Cortrium
Apex

Welcome

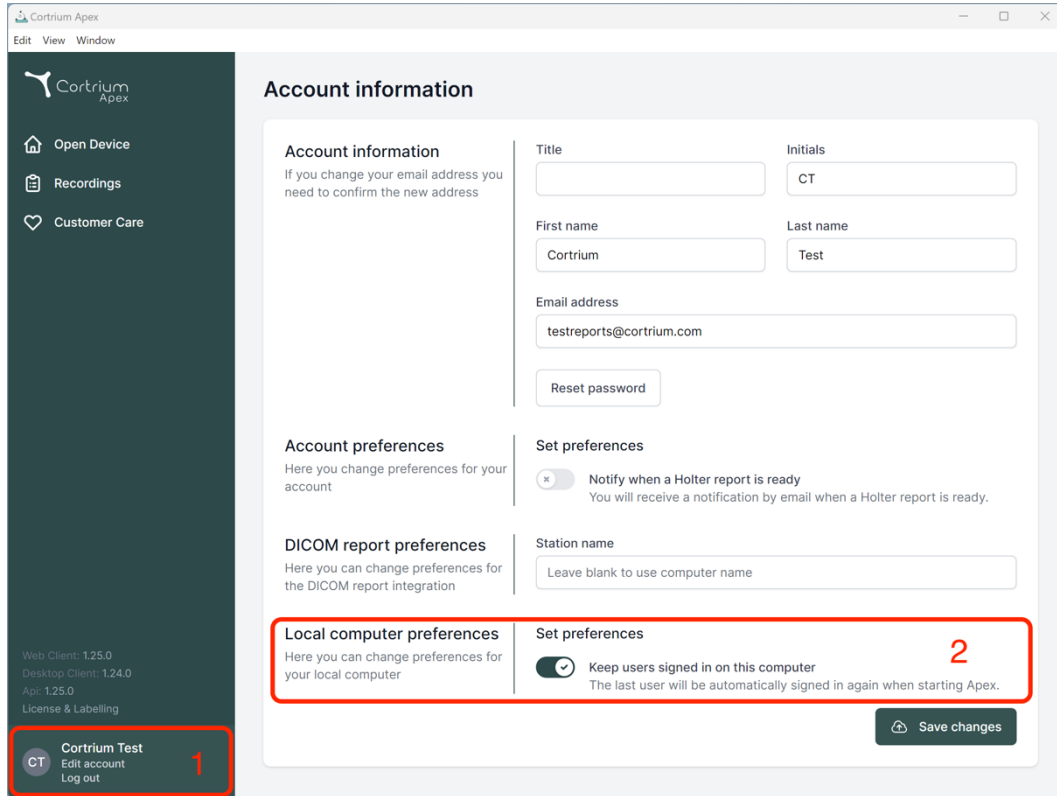
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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Account information

Account information
If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name:

Local computer preferences
Here you can change preferences for your local computer

☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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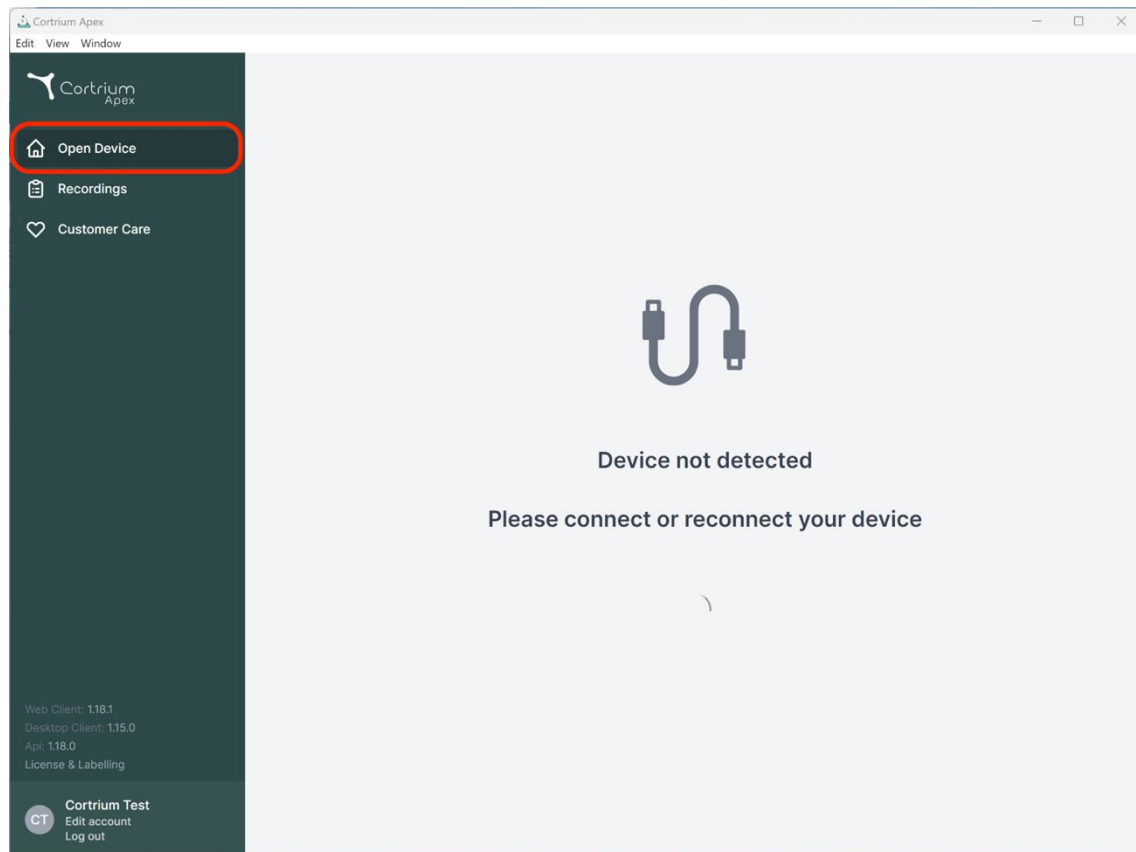
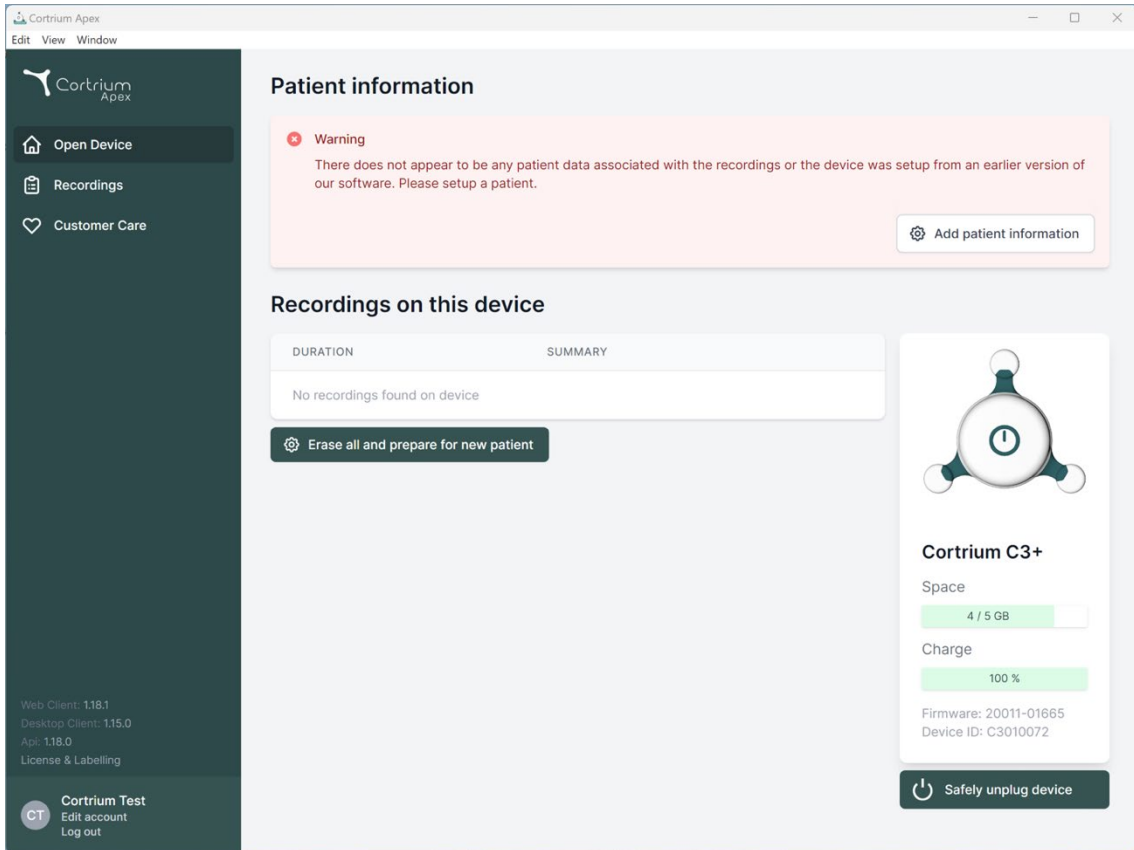
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area is titled 'Patient information' and features a red warning box stating: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table header for 'DURATION' and 'SUMMARY', showing 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' displays 'Space' (4 / 5 GB) and 'Charge' (100 %) with progress bars, and provides 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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Save to device

Warning This will erase all recordings on the device

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
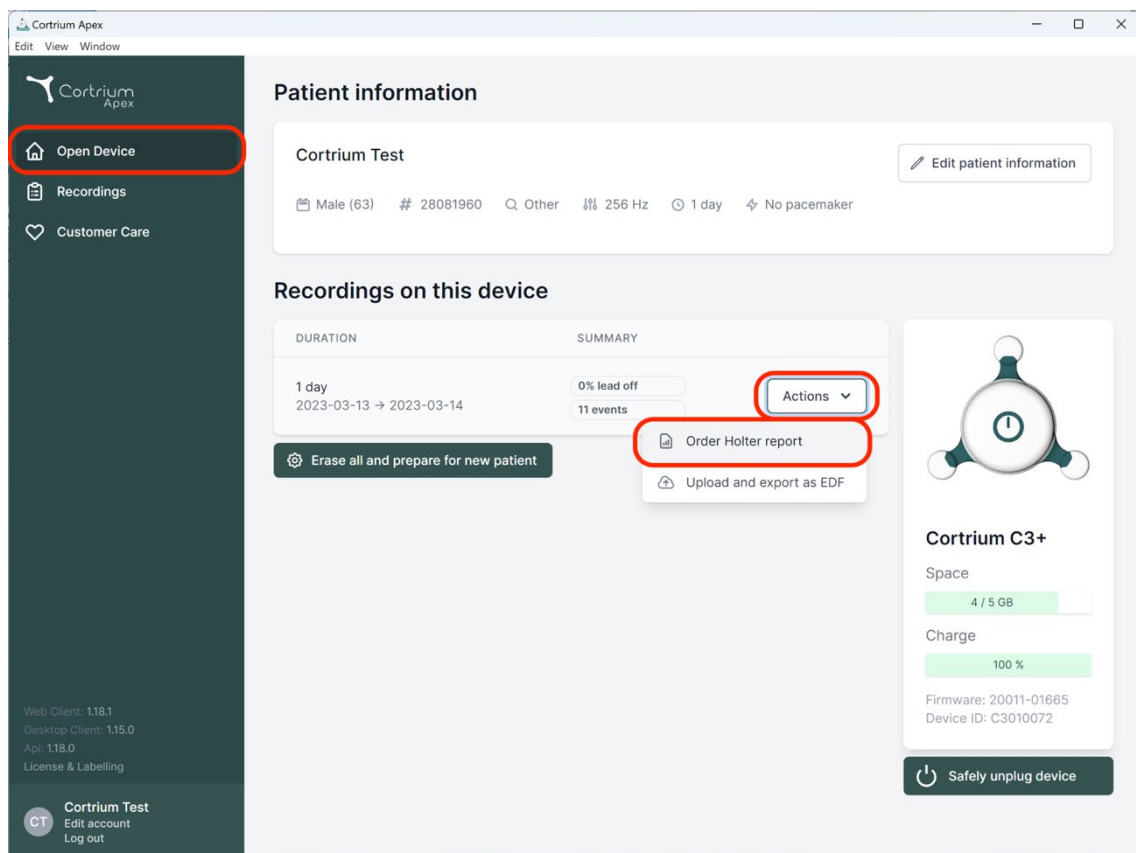
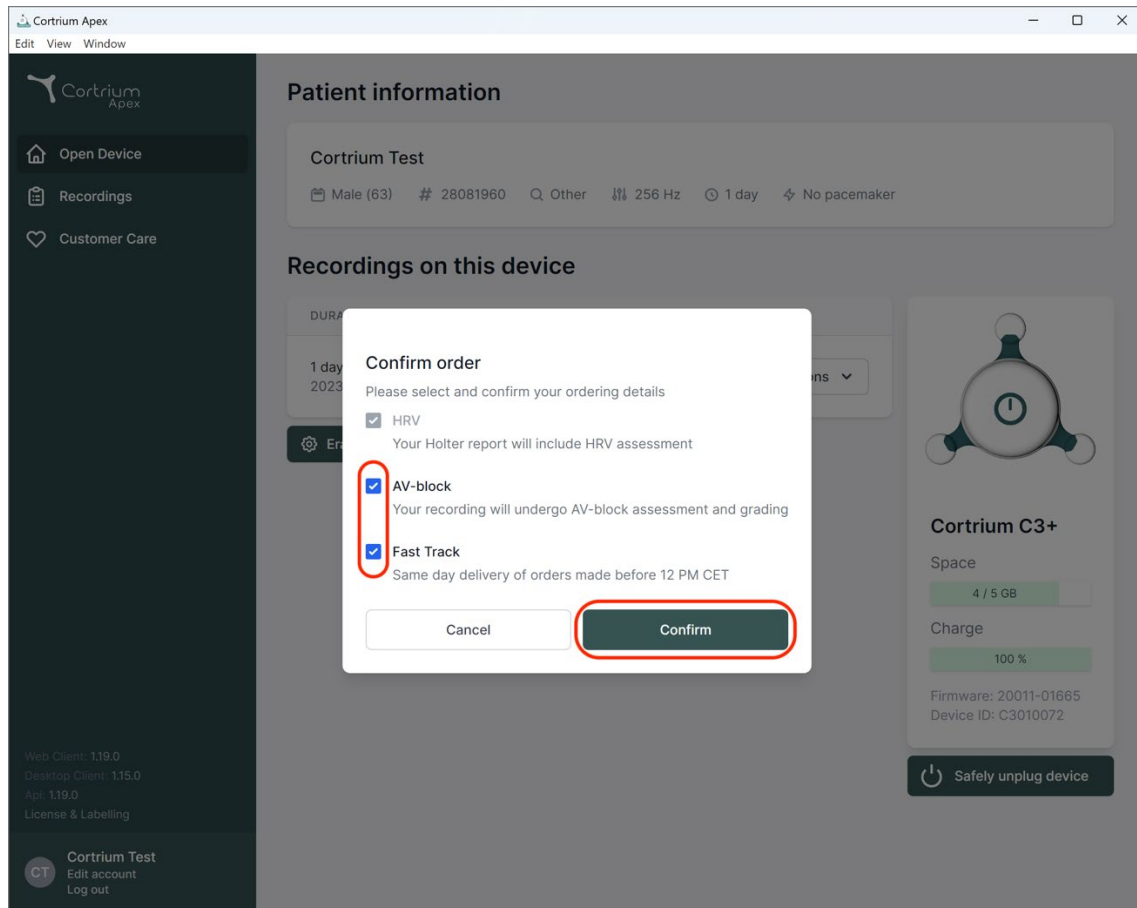
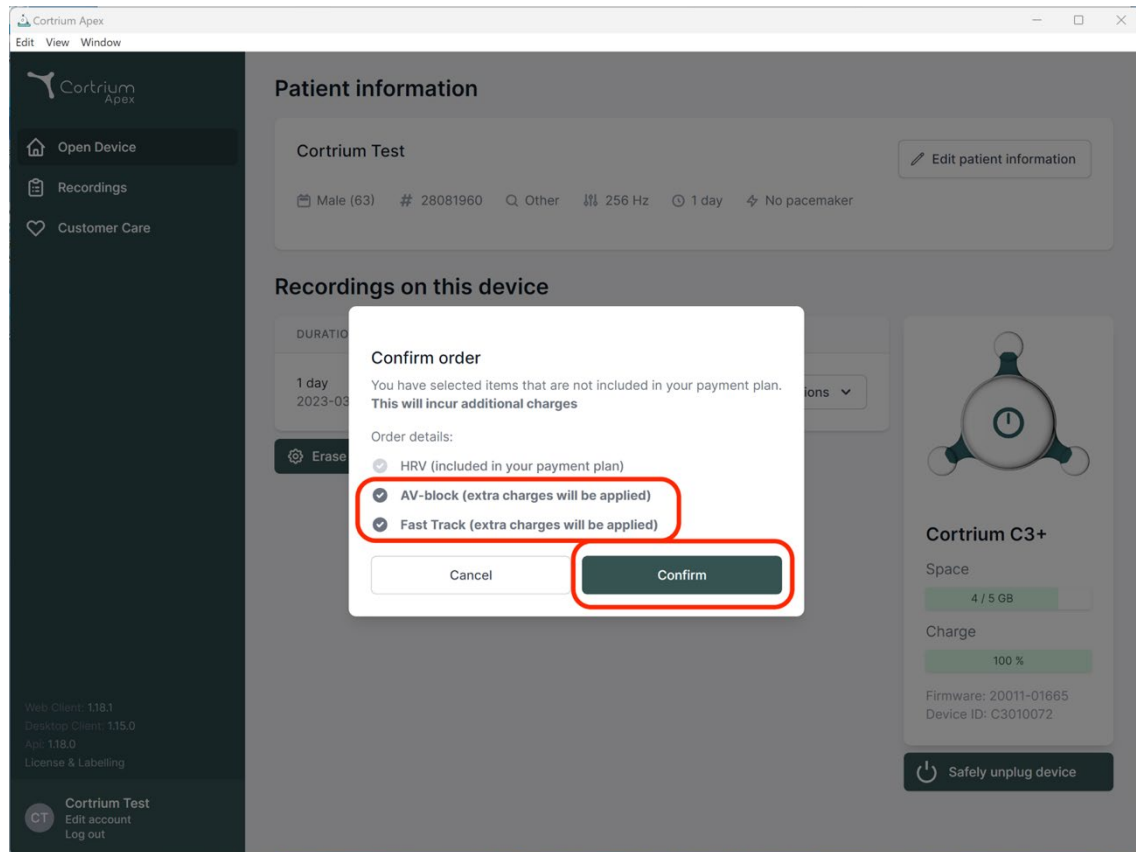
Herefter uploades optagelsen.

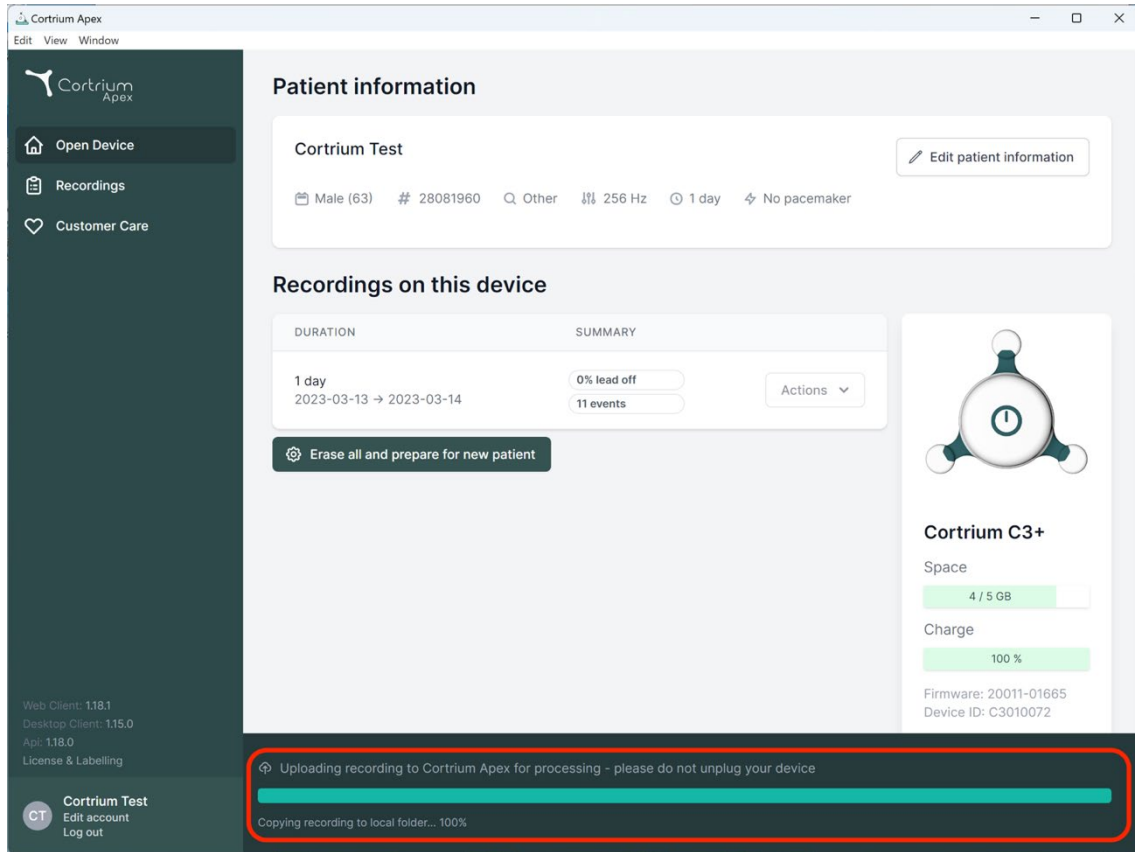The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'Api: 1.18.0', and 'License & Labelling'. At the bottom of the sidebar is a user profile for 'Cortrium Test' with an 'Edit account' and 'Log out' option.

The main content area is titled 'Patient information' and displays data for 'Cortrium Test'. It includes fields for 'Male (63)', '28081960', 'Other', '256 Hz', '1 day', and 'No pacemaker'. An 'Edit patient information' button is in the top right.

Below this is the 'Recordings on this device' section. It features a table with columns 'DURATION' and 'SUMMARY'. The table shows a recording for '1 day' from '2023-03-13' to '2023-03-14' with '0% lead off' and '11 events'. An 'Actions' dropdown menu is next to it. Below the table is a button that says 'Erase all and prepare for new patient'.

On the right side of the recording section is a device status card for 'Cortrium C3+'. It shows 'Space' at '4 / 5 GB' and 'Charge' at '100 %'. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'.

At the bottom of the interface, a red-bordered box highlights a status bar. It contains the text 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and a progress bar. Below the progress bar, it says 'Copying recording to local folder... 100%'.

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
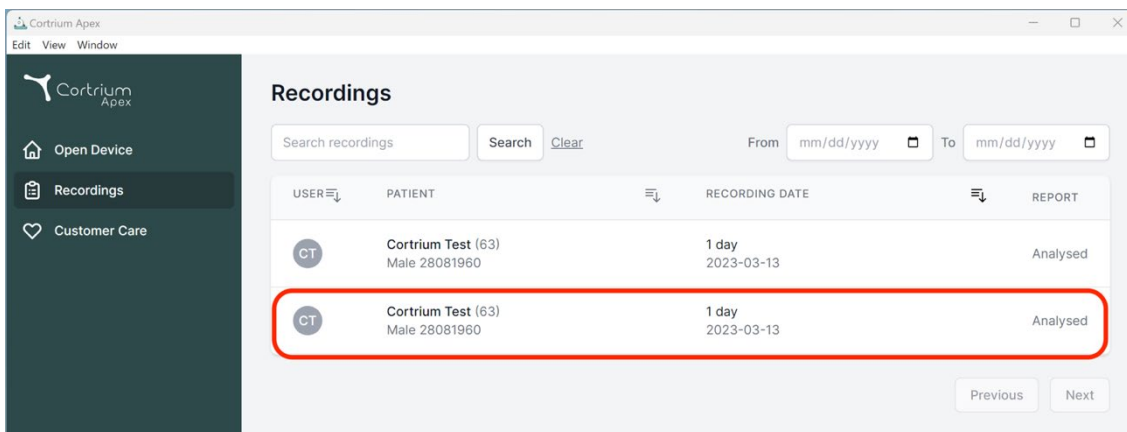
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

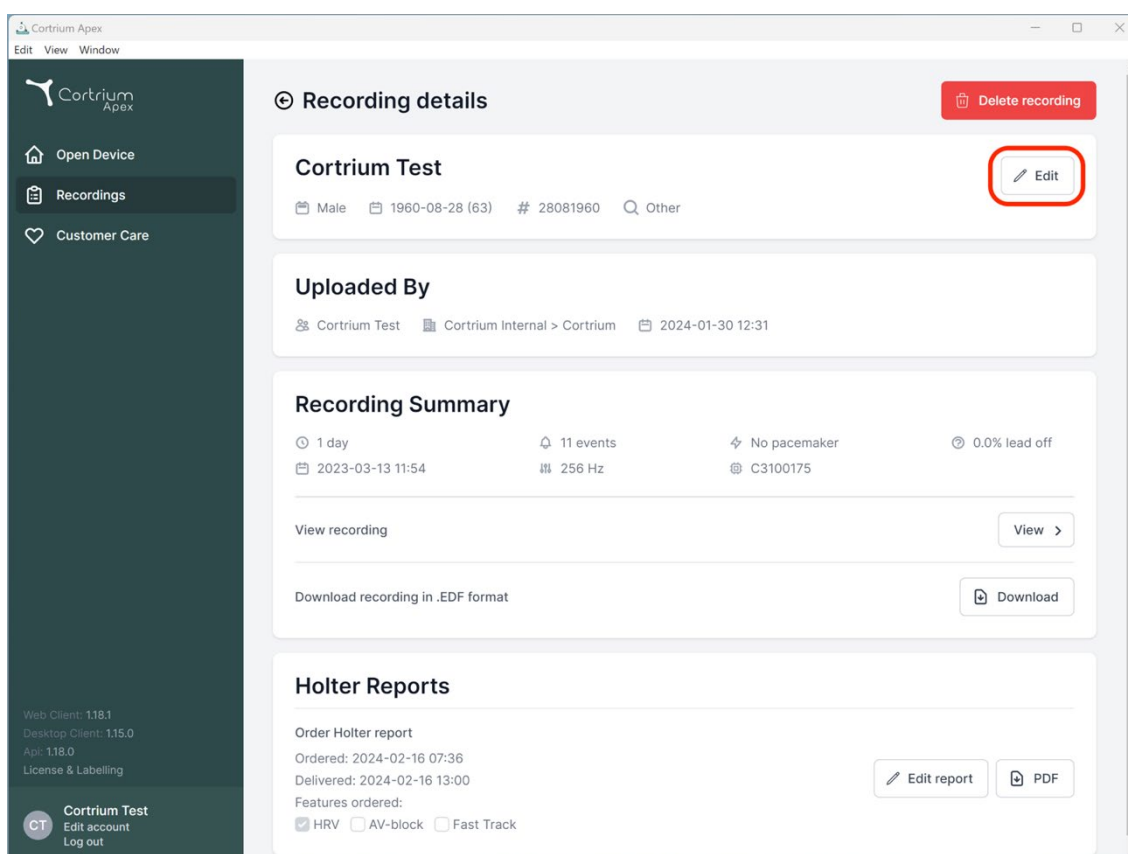Patientoplysningerne kan redigeres ved at klikke på knappen "Rediger".

Recording details

Cortrium Test

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Order Holter report
Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00
Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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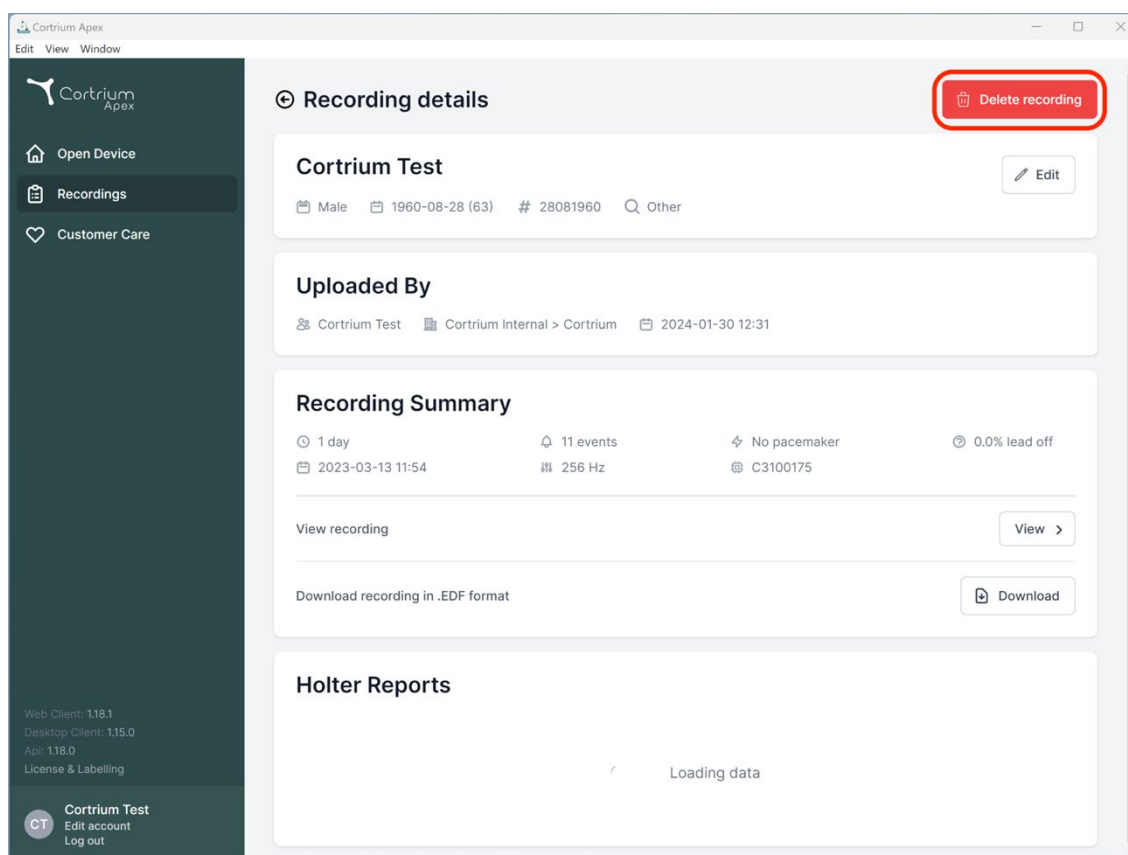
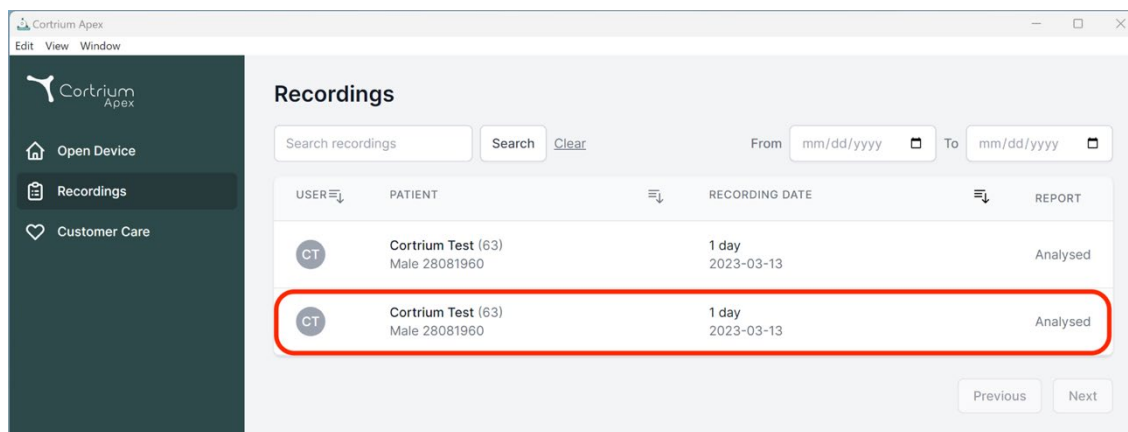
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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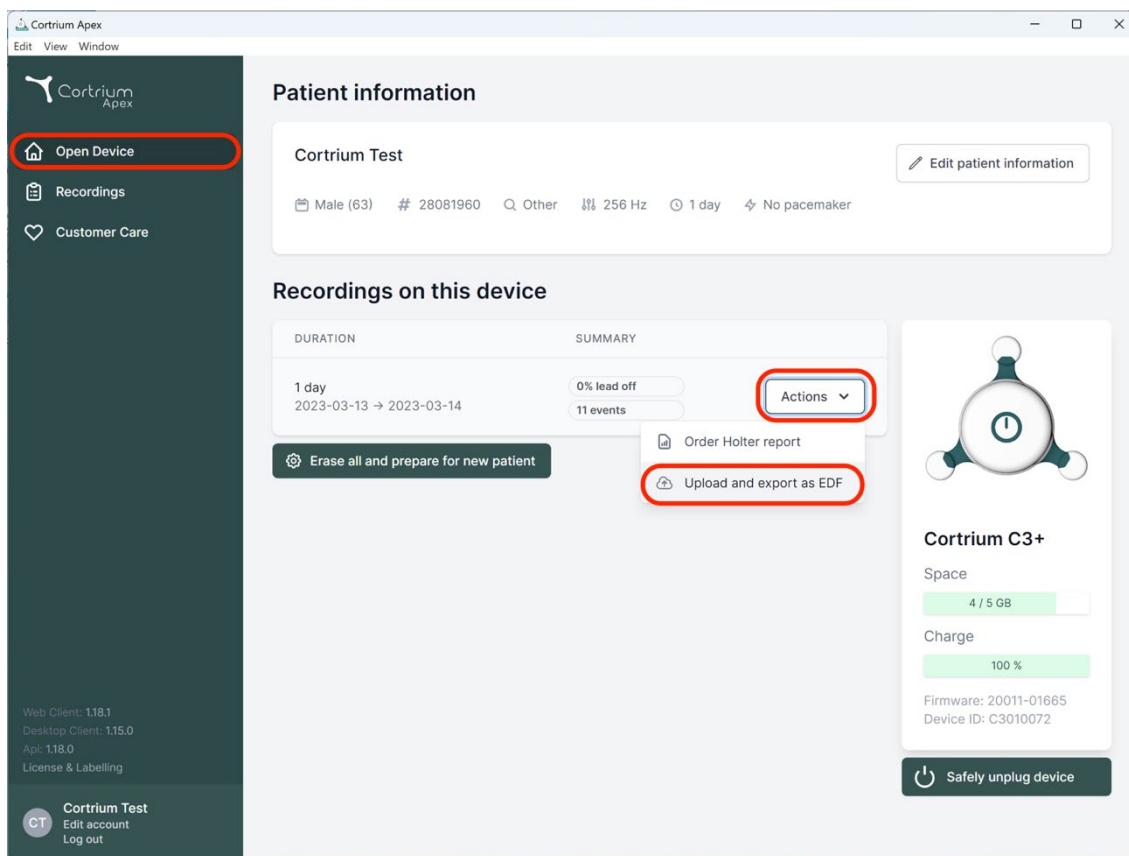
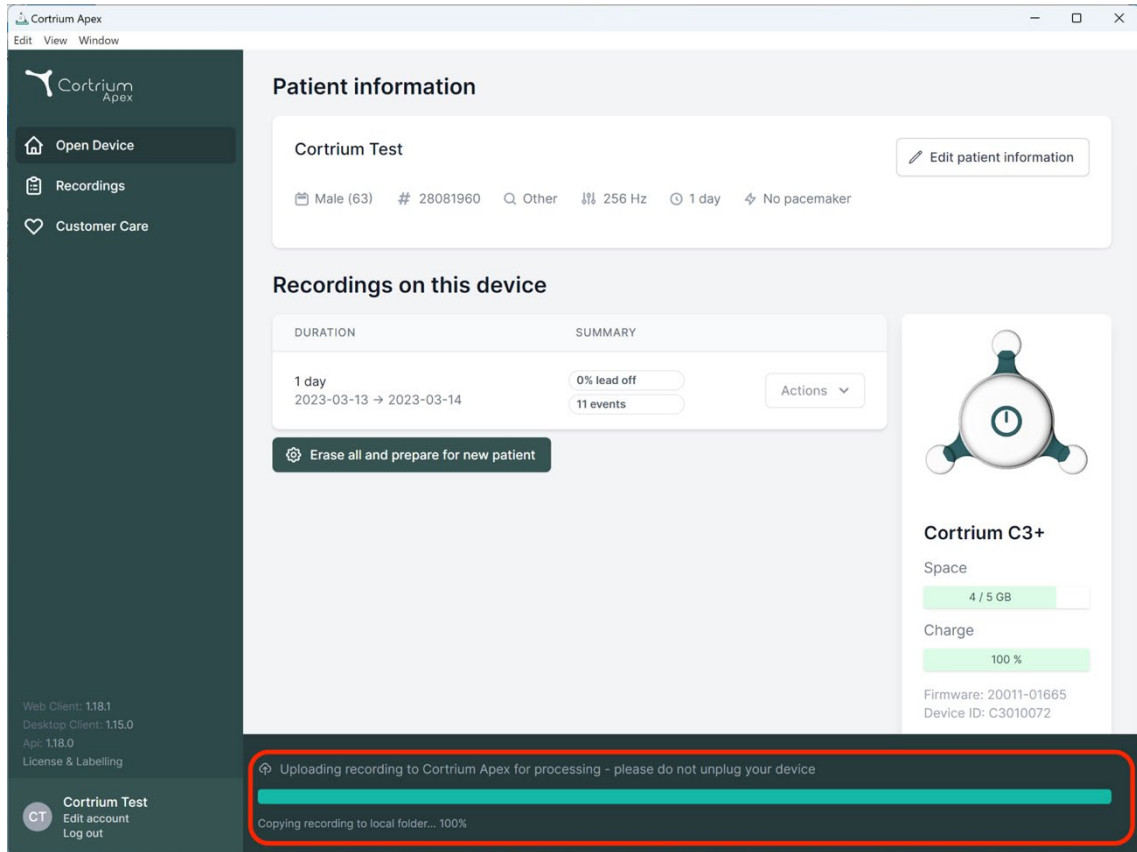
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

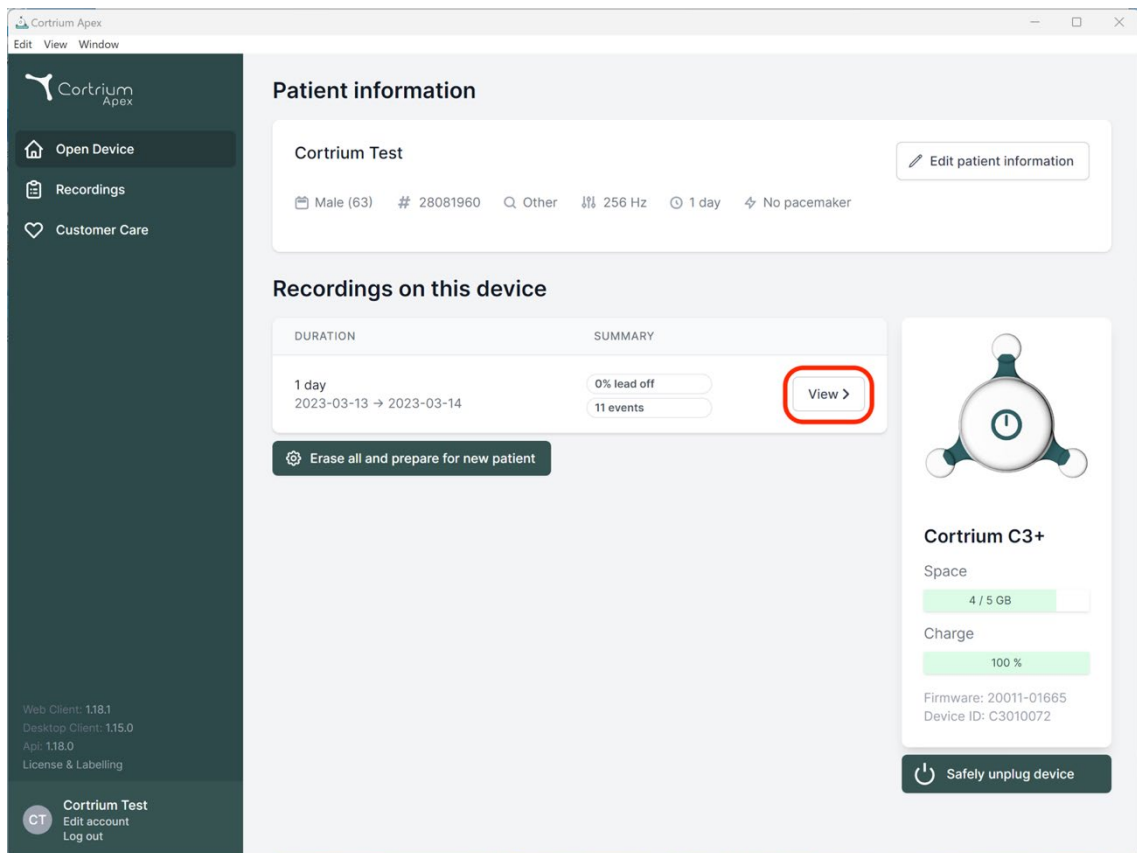
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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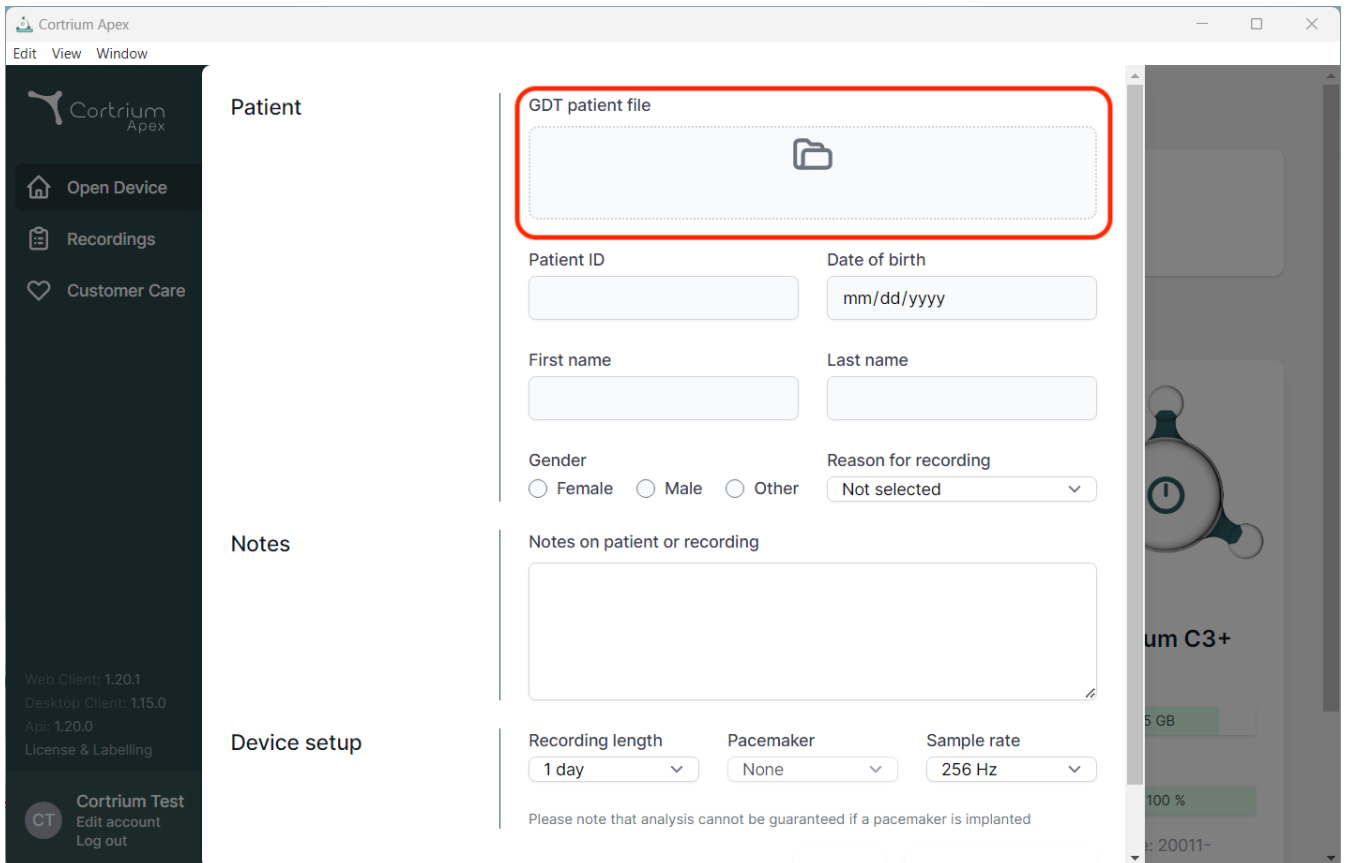
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a form for entering patient information. The GDT patient file field is highlighted with a red box. Below it are fields for Patient ID, Date of birth, First name, Last name, Gender, and Reason for recording. The Notes section contains a text area for notes on the patient or recording. The Device setup section contains fields for Recording length, Pacemaker, and Sample rate.

Patient

GDT patient file

Patient ID

Date of birth

First name

Last name

Gender

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length

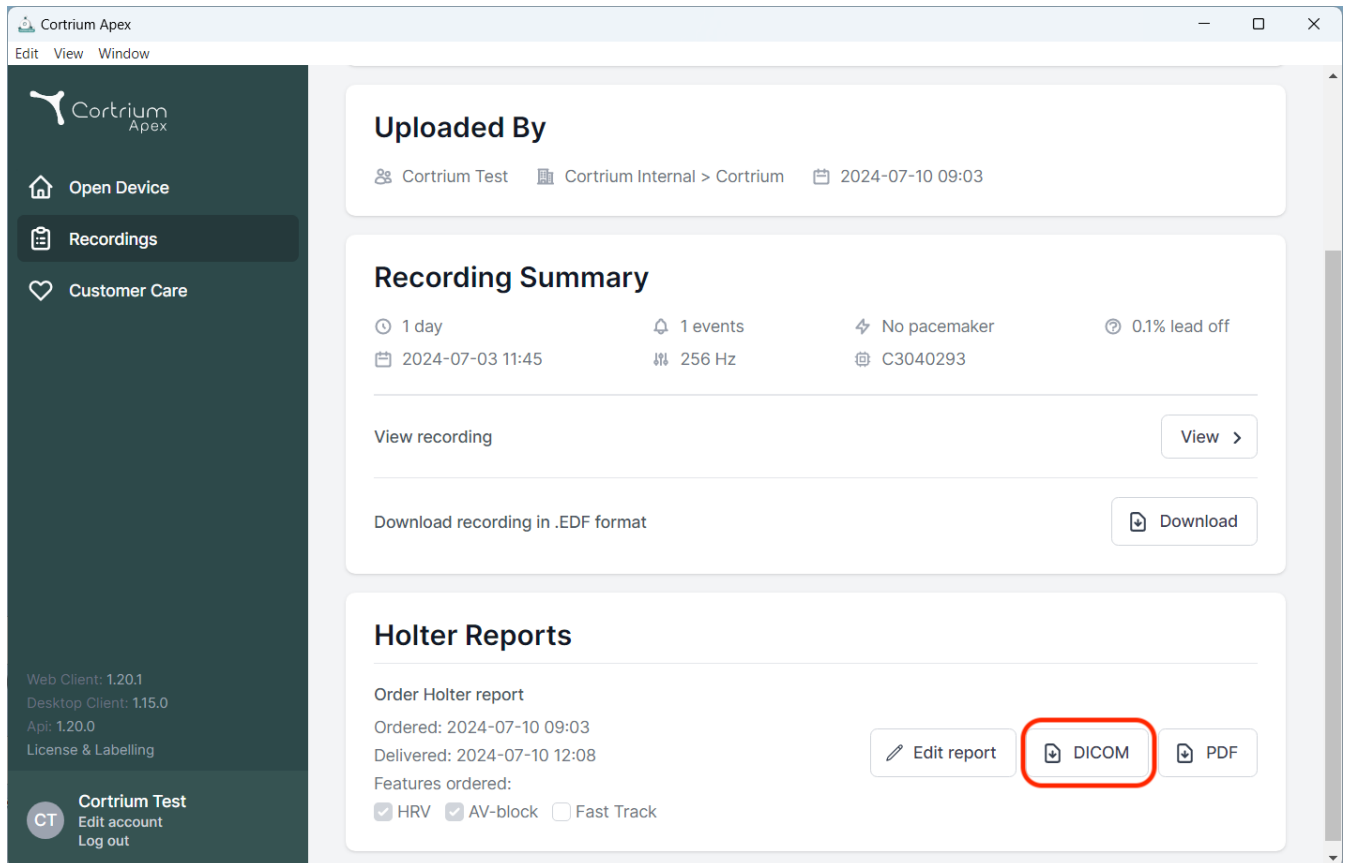
Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test', 'Cortrium Internal > Cortrium', and the date '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', 'C3040293', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (for .EDF format).
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

At the bottom left of the sidebar, version information is listed: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'Api: 1.20.0', and 'License & Labelling'. A user profile for 'Cortrium Test' is also visible.

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Account information

Account information

If you change your email address you need to confirm the new address

Title

Initials

First name

Last name

Email address

Reset password

Account preferences

Here you change preferences for your account

Set preferences

☐

Notify when a Holter report is ready

You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

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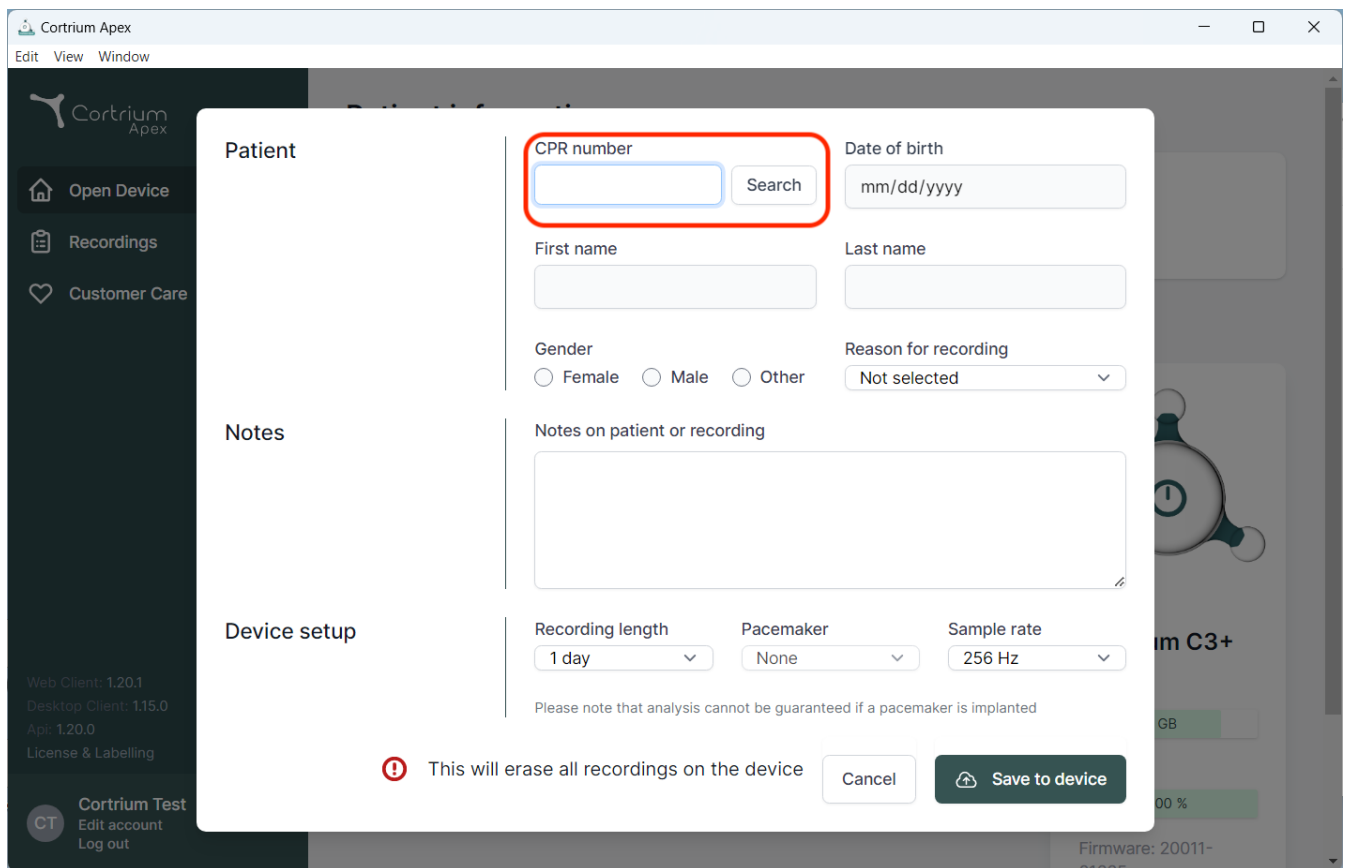
The screenshot shows the Cortrium Apex software interface. A modal window is open for patient information. The 'Patient' section contains the following fields:

- CPR number:** A text input field with a 'Search' button next to it. This field is highlighted with a red box.
- Date of birth:** A text input field with the placeholder 'mm/dd/yyyy'.
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for 'Female', 'Male', and 'Other'.
- Reason for recording:** A dropdown menu currently showing 'Not selected'.

The 'Notes' section contains a large text area labeled 'Notes on patient or recording'.

The 'Device setup' section contains the following fields:

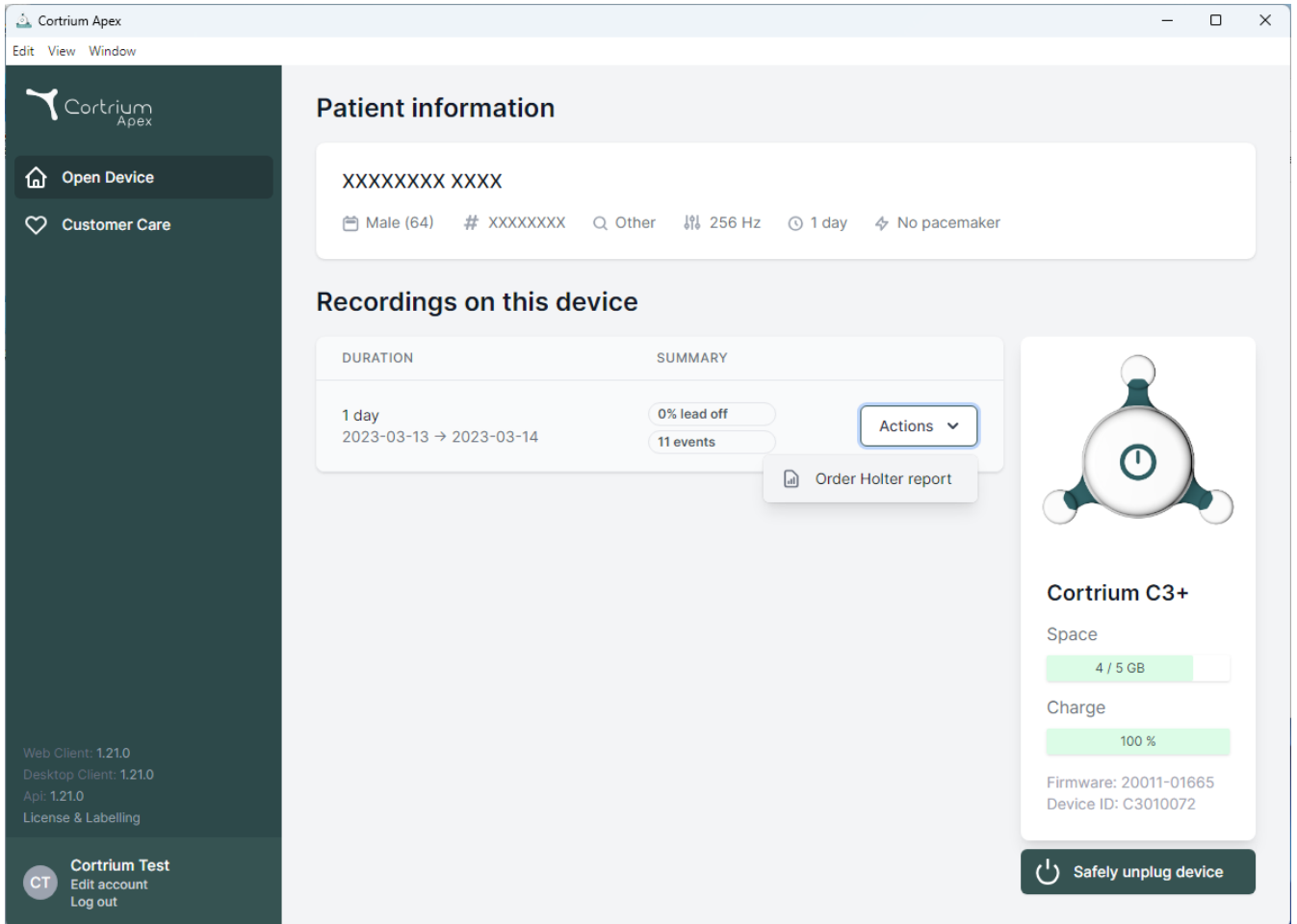
- Recording length:** A dropdown menu showing '1 day'.
- Pacemaker:** A dropdown menu showing 'None'.
- Sample rate:** A dropdown menu showing '256 Hz'.

Below the device setup fields, there is a warning message: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom of the modal, there is a red warning icon and the text: "This will erase all recordings on the device". To the right of this message are two buttons: 'Cancel' and 'Save to device'.

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device" and "Customer Care". The main content area is divided into two sections: "Patient information" and "Recordings on this device".

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Order Holter report

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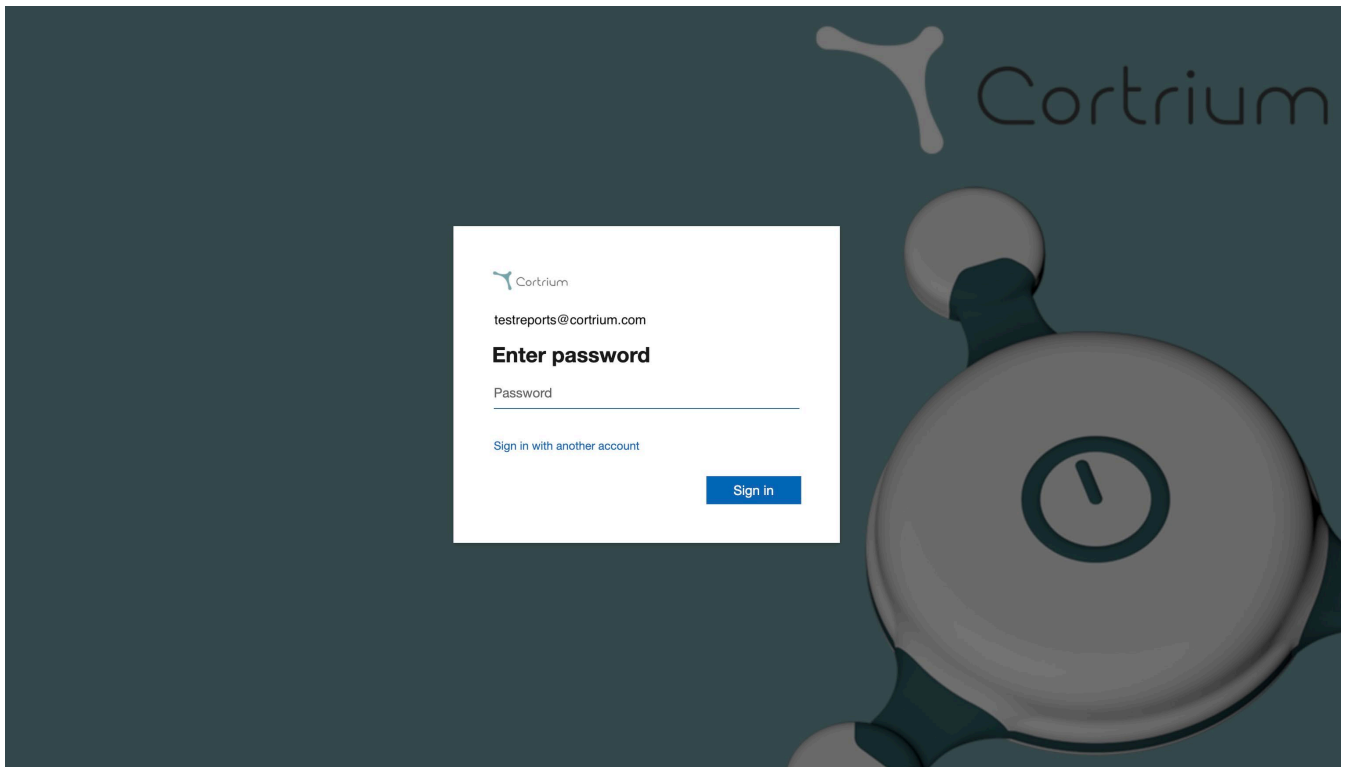
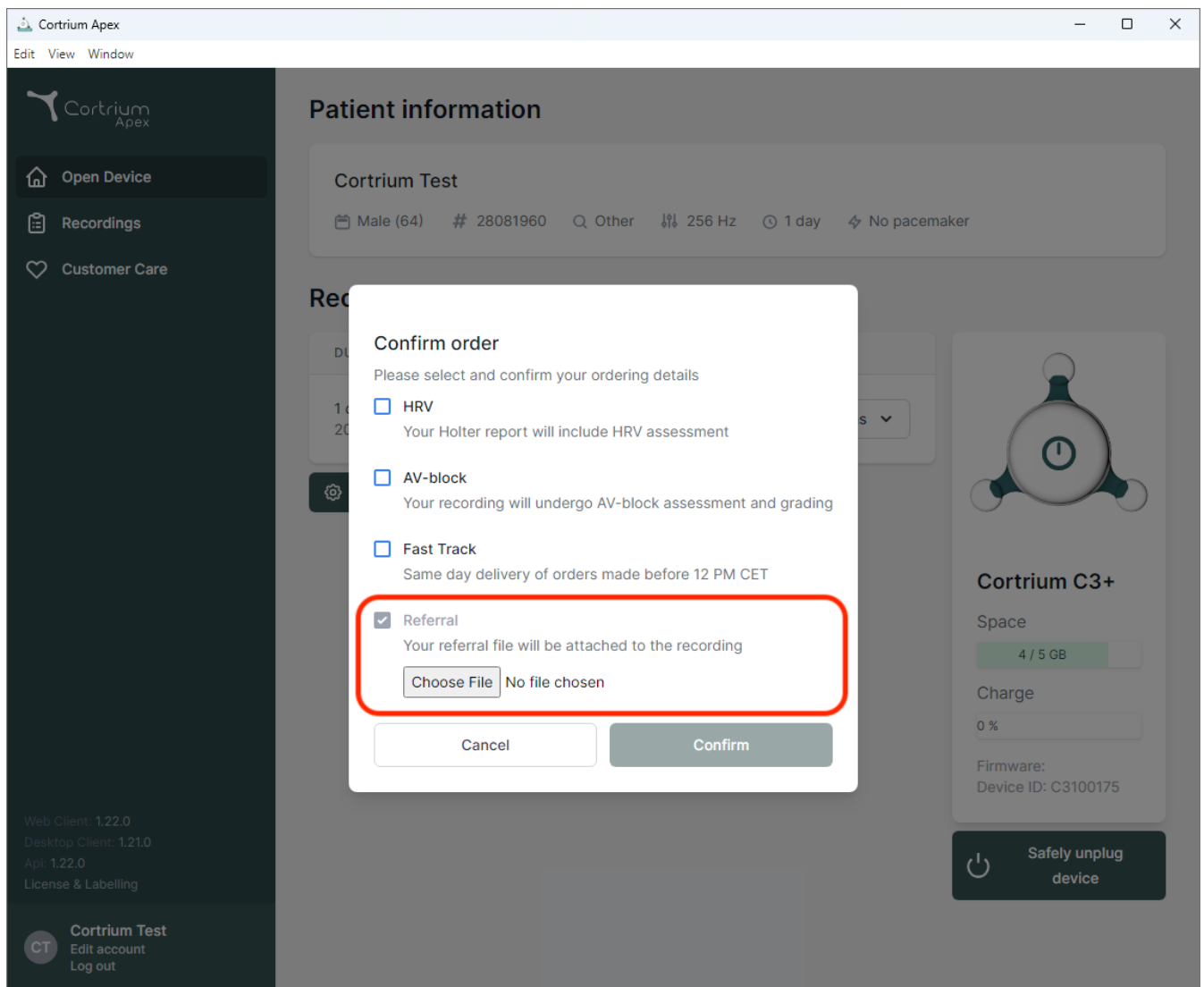
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
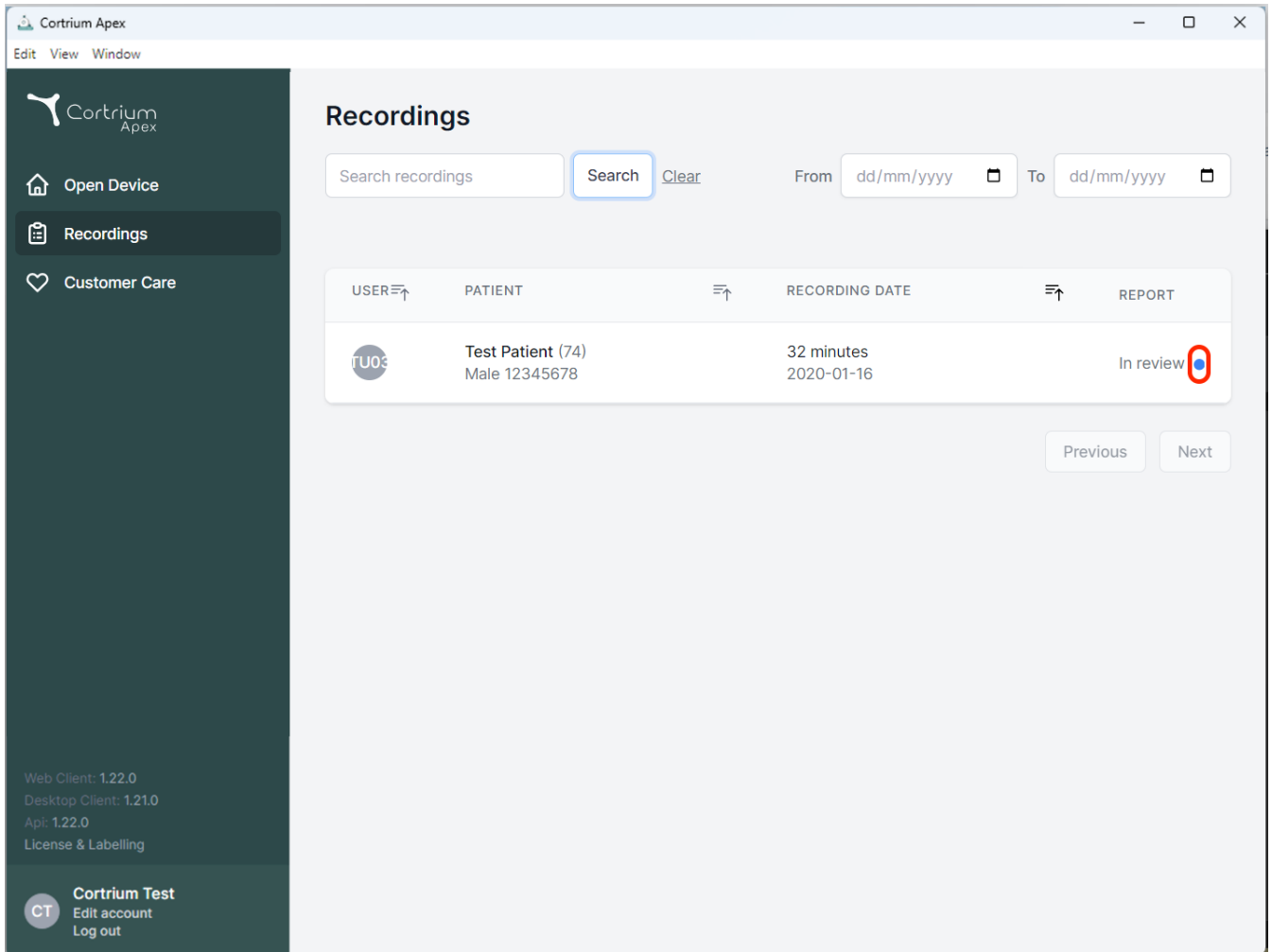
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area is titled "Recordings" and includes a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a status of "In review". A blue dot is visible next to the "In review" status. At the bottom of the table are "Previous" and "Next" buttons. The footer of the sidebar shows version information and a user profile for "Cortrium Test".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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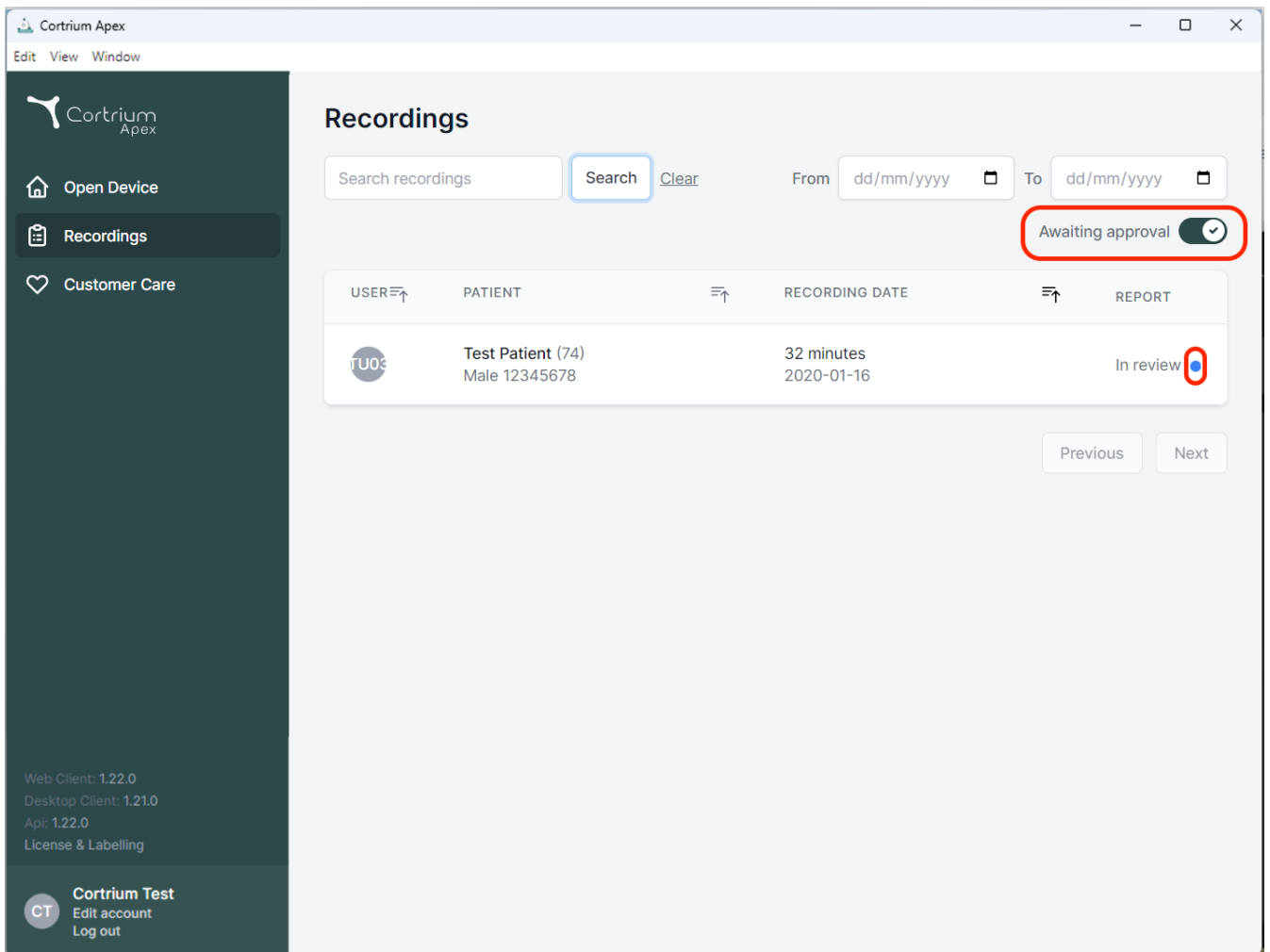
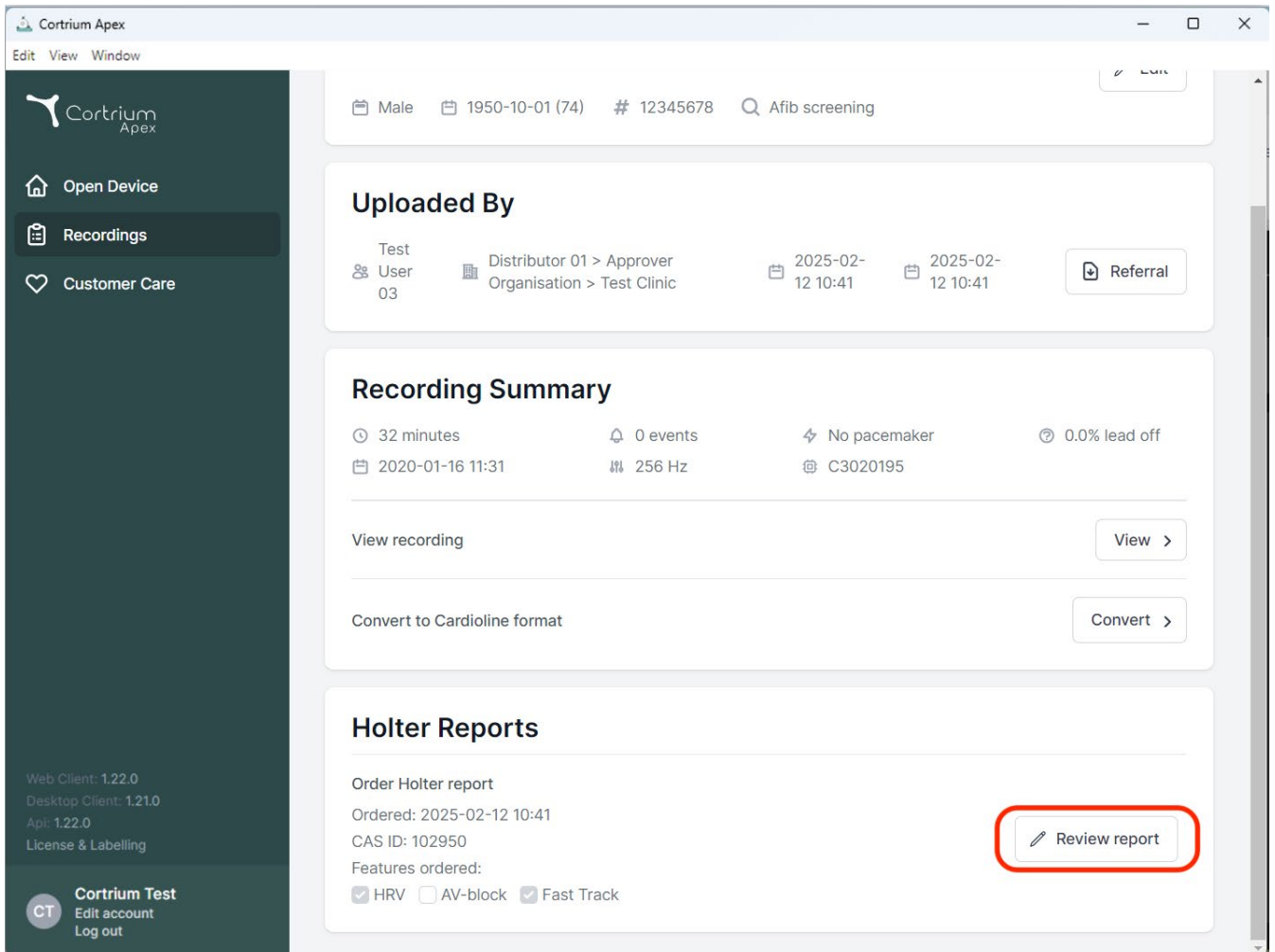
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main content area shows patient information at the top: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the "Uploaded By" section, showing Test User 03, Distributor 01 > Approver Organisation > Test Clinic, and upload dates of 2025-02-12 10:41. A "Referral" button is present. The "Recording Summary" section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It includes buttons for "View recording" and "Convert to Cardioline format". The "Holter Reports" section shows an "Order Holter report" with details: Ordered: 2025-02-12 10:41, CAS ID: 102950, and Features ordered: HRV, AV-block, and Fast Track. A "Review report" button is highlighted with a red rectangle.

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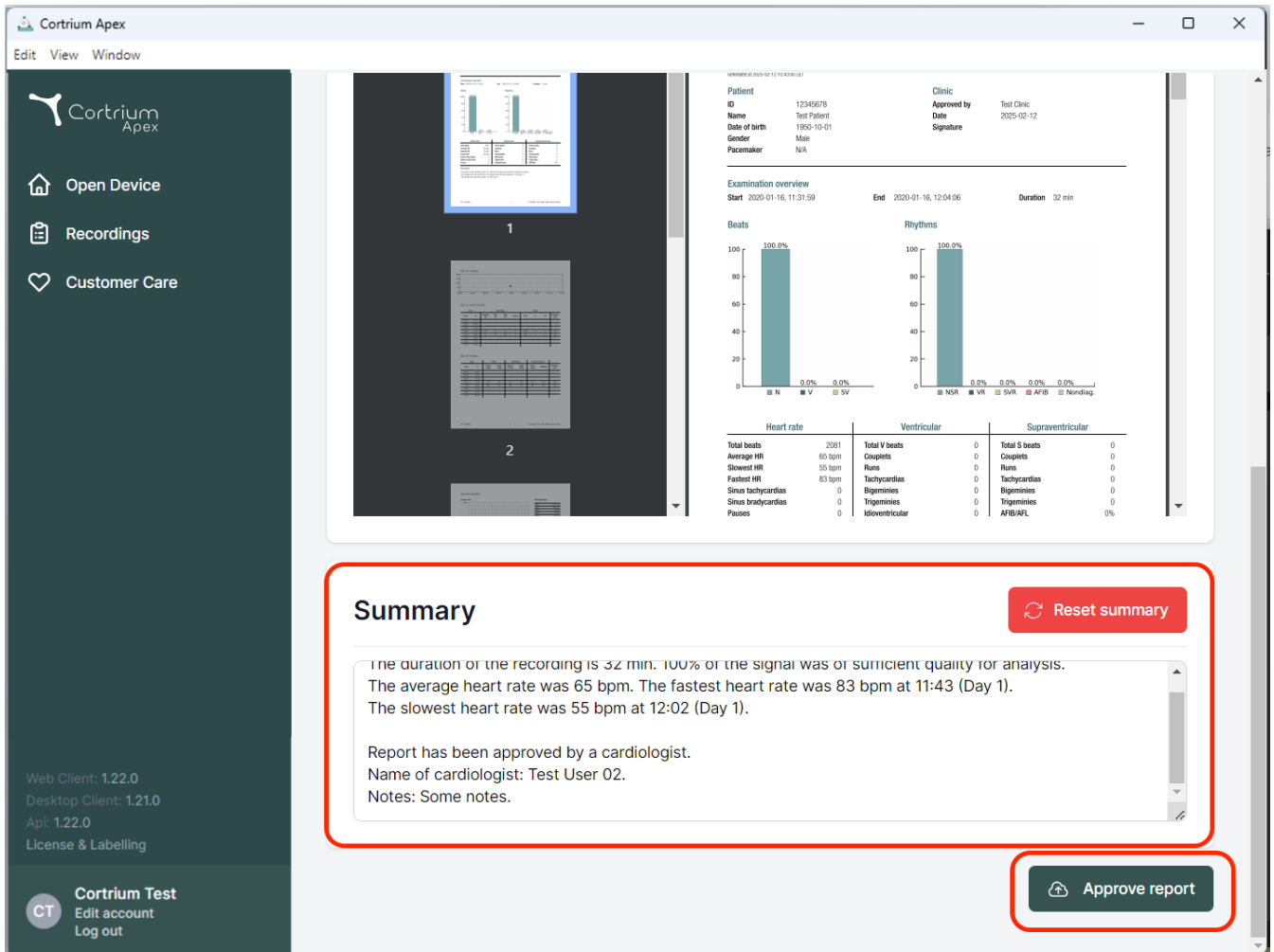Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient
ID: 12345678
Name: Test Patient
Date of birth: 1950-10-01
Gender: Male
Pacemaker: N/A

Clinic
Approved by: Test Clinic
Date: 2025-02-12
Signature:

Examination overview
Start: 2020-01-16, 11:31:59
End: 2020-01-16, 12:04:06
Duration: 32 min

Beats
100.0%
0.0% 0.0% 0.0%

Rhythms
100.0%
0.0% 0.0% 0.0% 0.0% 0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Idioventricular	0	AFIB/AF	0%

Summary Reset summary

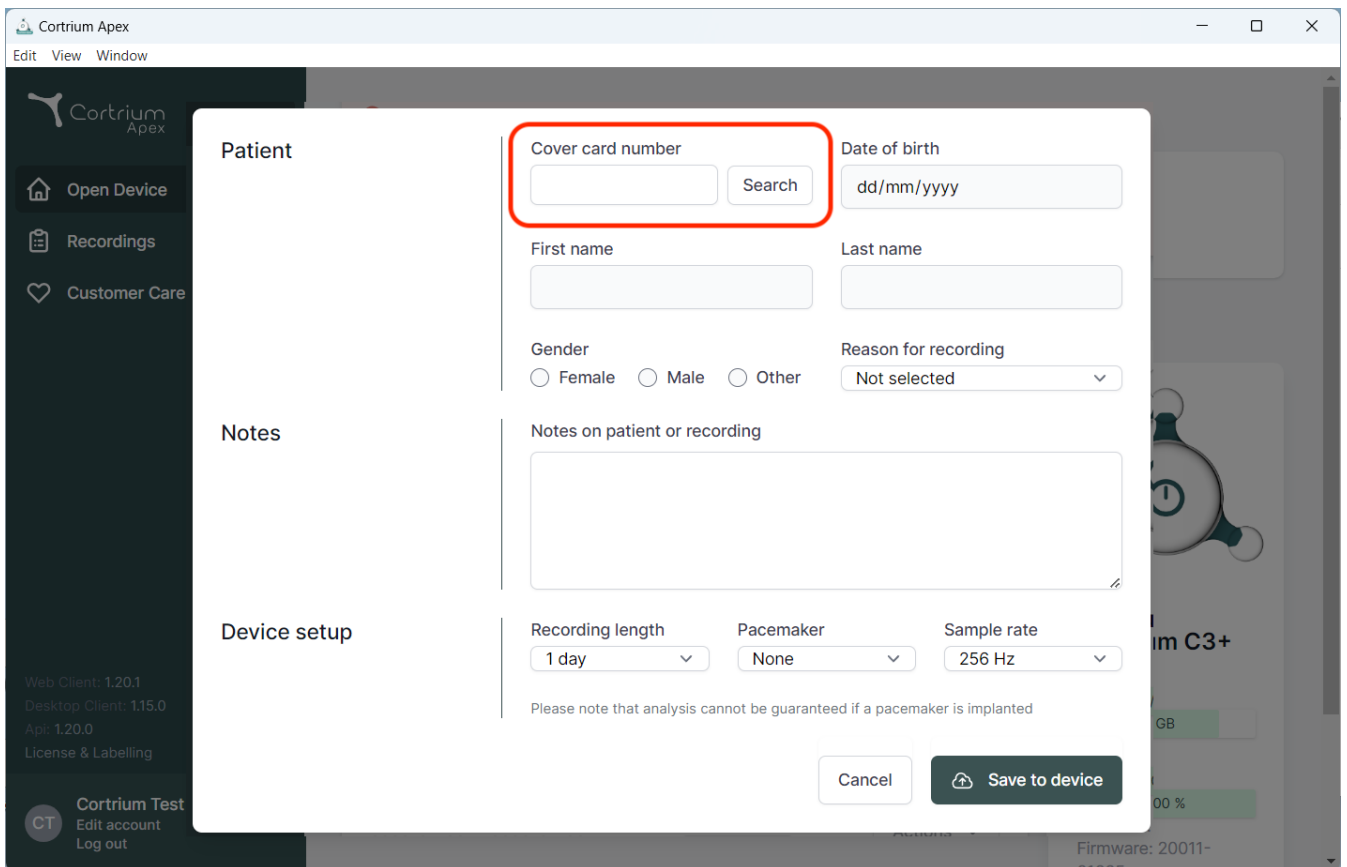
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying a form for patient information. The form includes fields for "Cover card number" (highlighted with a red box), "Date of birth" (format dd/mm/yyyy), "First name", "Last name", "Gender" (radio buttons for Female, Male, Other), and "Reason for recording" (dropdown menu, currently "Not selected"). There is also a large text area for "Notes on patient or recording". Below the form, there are dropdown menus for "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom right of the form are "Cancel" and "Save to device" buttons. The background shows the main interface with a sidebar containing "Open Device", "Recordings", and "Customer Care" options, and a top menu with "Edit", "View", and "Window".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark
E-mail: info@cortrium.com • Hjemmeside: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47