Cortrium Apex 1.27

Cortrium Apex 1.27

Kasutusjuhend

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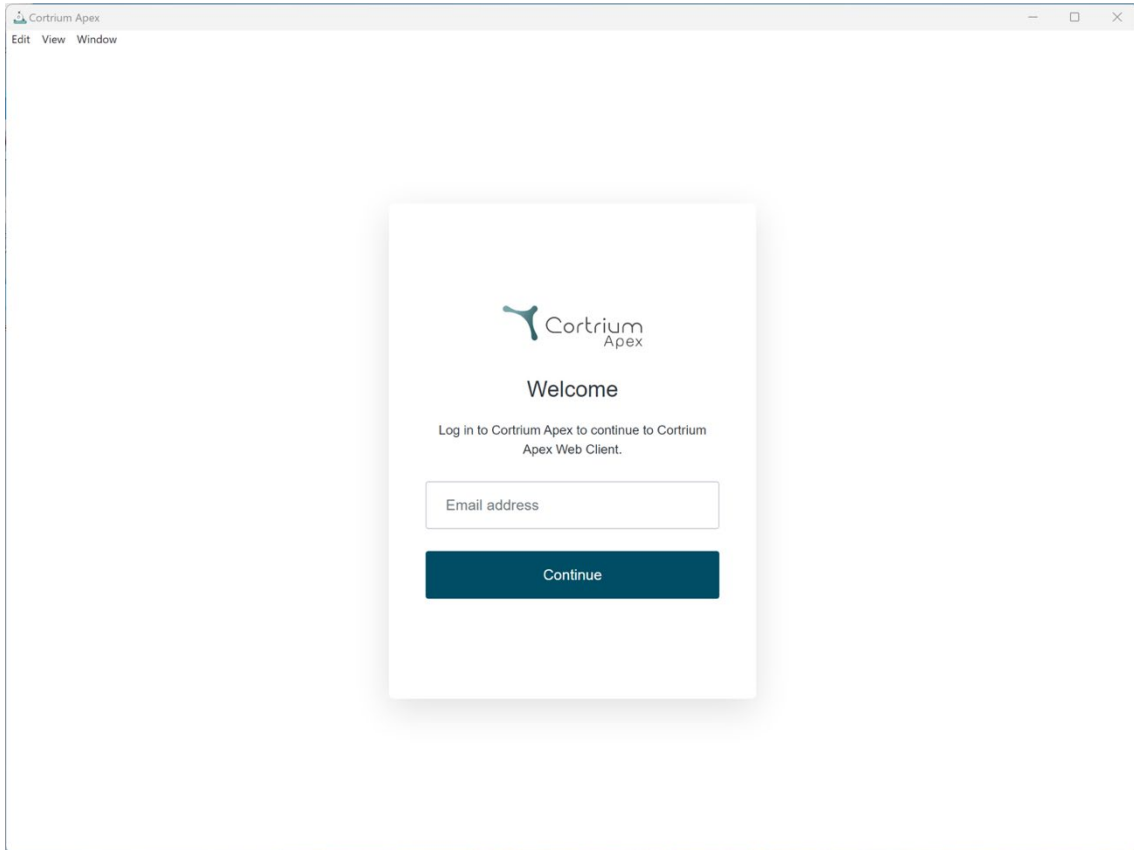
 Cortrium
Apex

Welcome

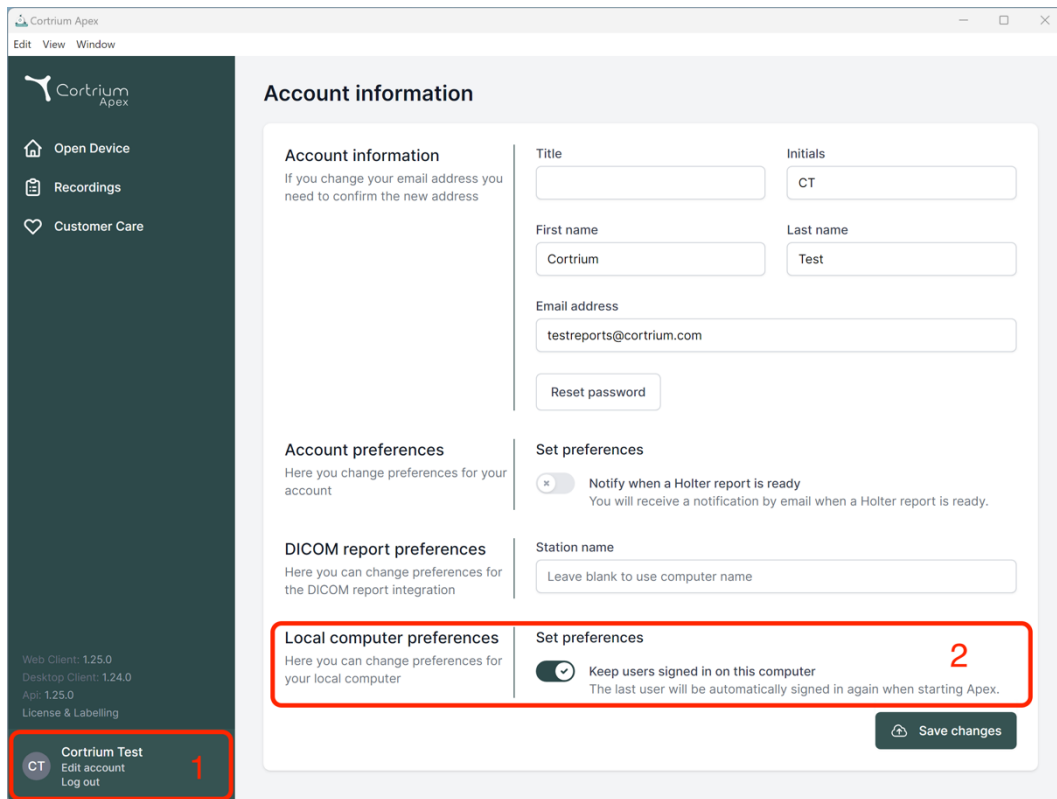
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.25.0, Desktop Client: 1.24.0, Api: 1.25.0, License & Labelling) and a user profile section for 'Cortrium Test' with options to 'Edit account' and 'Log out'. This sidebar section is highlighted with a red box and a red number '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' (with a note about email changes), 'Account preferences' (for account-level settings), 'DICOM report preferences' (for report integration), 'Local computer preferences' (for local device settings), and 'Set preferences' (for user-specific settings). The 'Local computer preferences' and 'Set preferences' sections are highlighted with a red box and a red number '2'. The 'Set preferences' section includes a toggle for 'Keep users signed in on this computer', which is currently turned on. A 'Save changes' button is located at the bottom right of the main content area.

Account information

If you change your email address you need to confirm the new address

Title: Initials:

First name: Last name:

Email address:

Account preferences

Here you change preferences for your account

DICOM report preferences

Here you can change preferences for the DICOM report integration

Local computer preferences

Here you can change preferences for your local computer

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

Station name

Local computer preferences

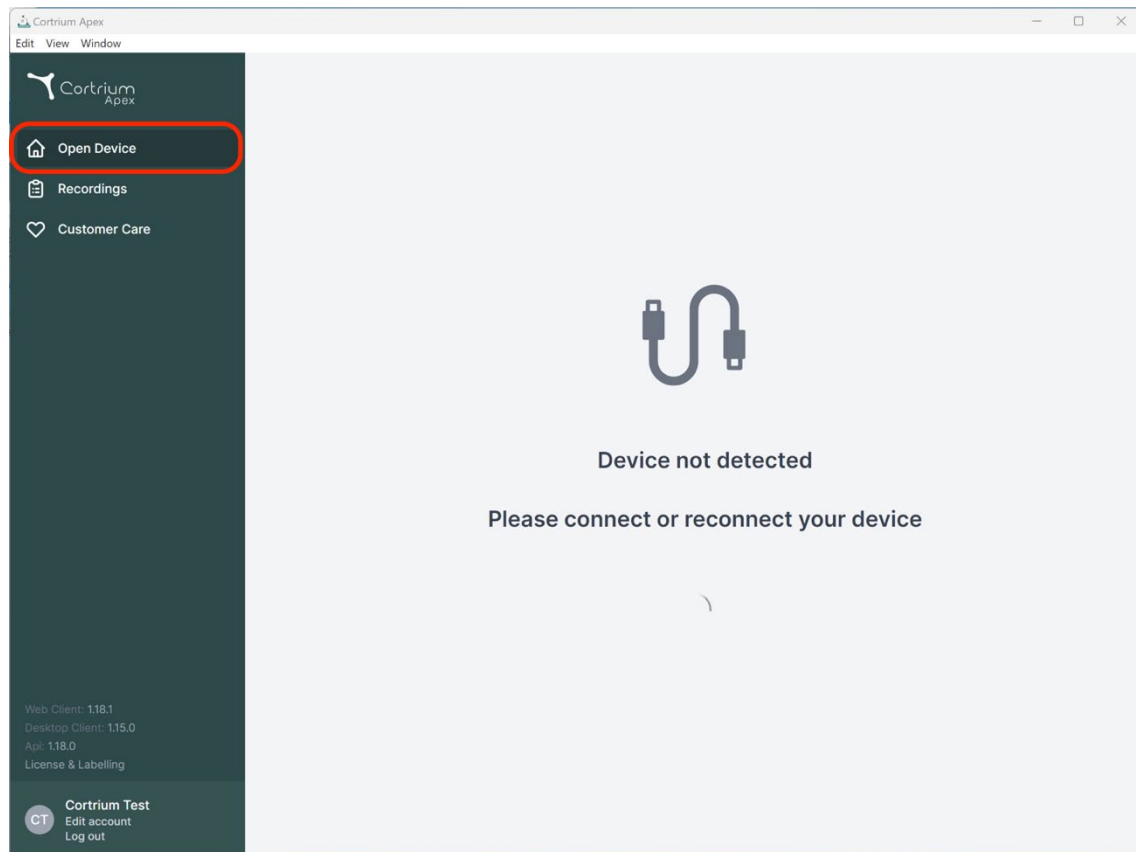
☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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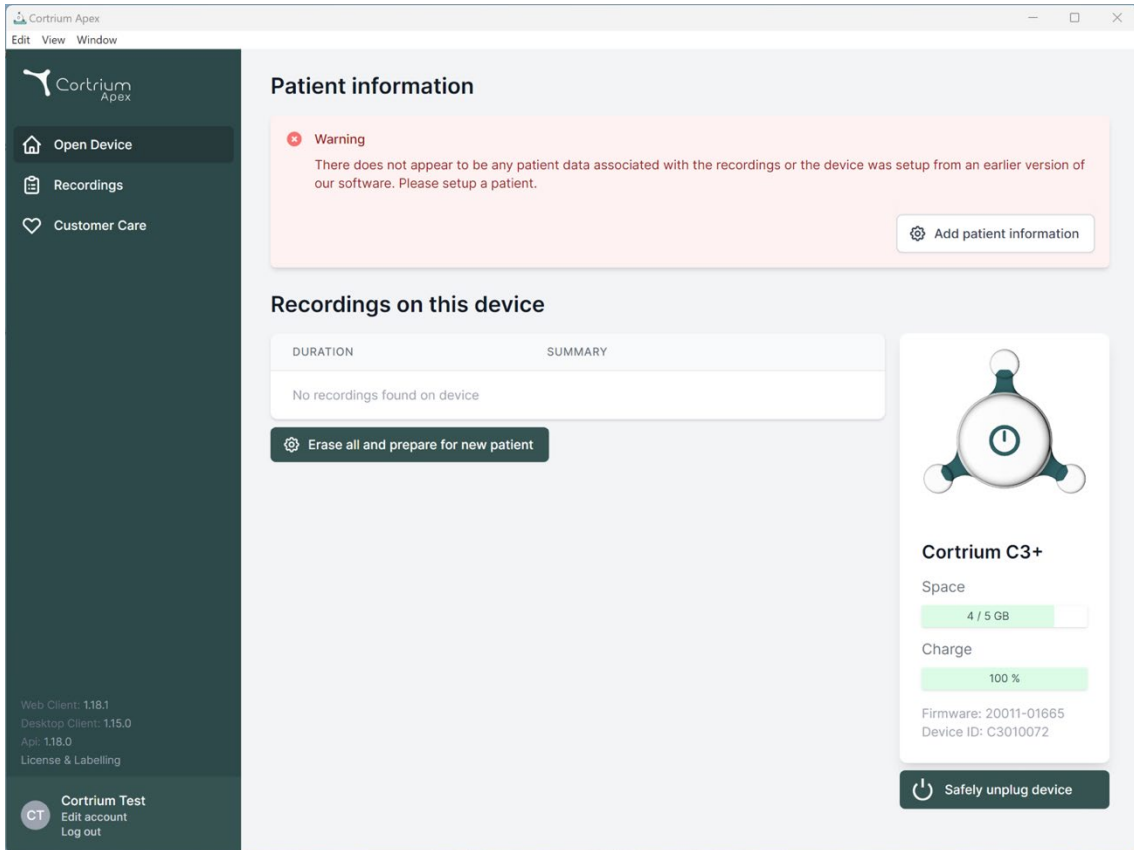
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' links. The main content area has a 'Patient information' section with a warning message: 'There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' and an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table showing 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right is a device status card for 'Cortrium C3+' showing 'Space' (4 / 5 GB) and 'Charge' (100 %) with progress bars, and a 'Safely unplug device' button at the bottom.

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Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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Save to device

Warning
This will erase all recordings on the device

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
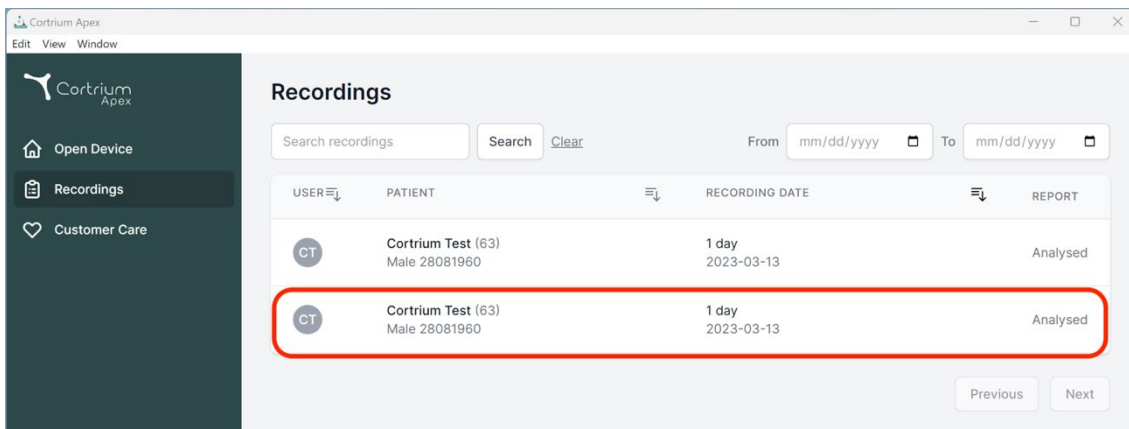
Recordings

Search recordings Search [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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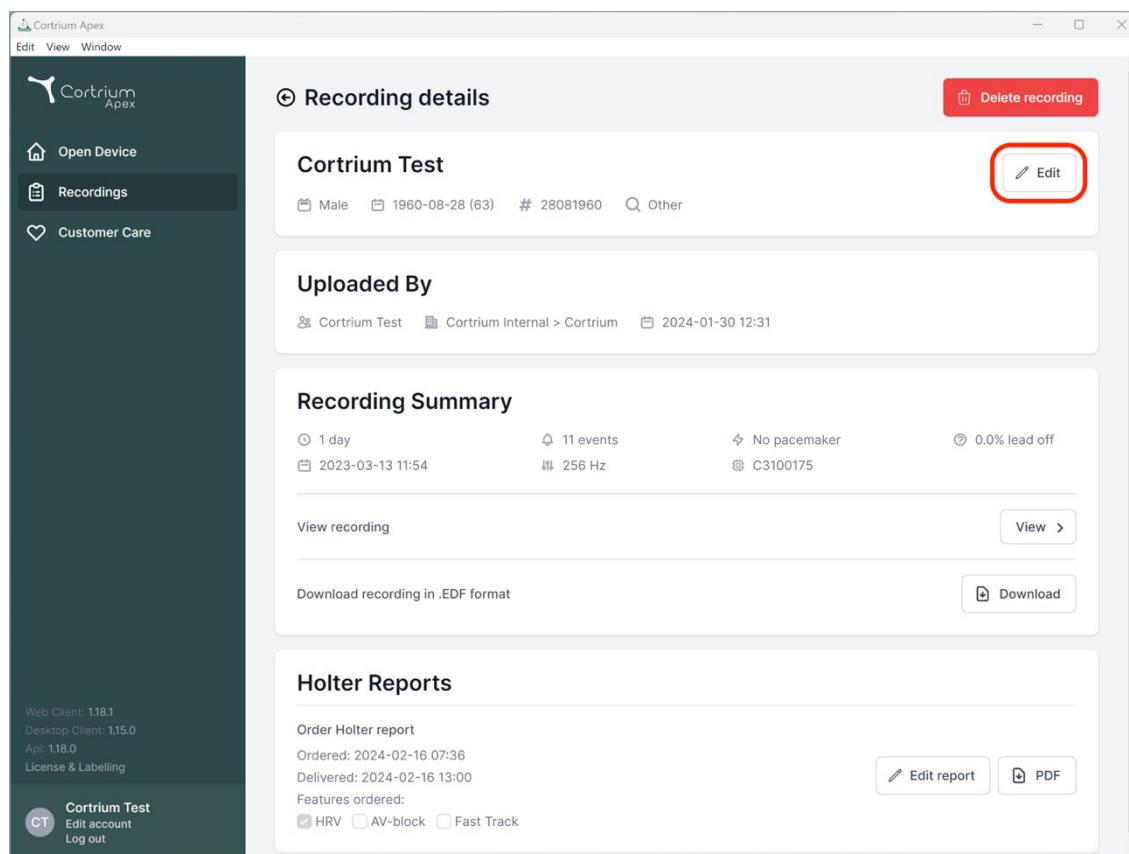Recording details [Delete recording](#)

Cortrium Test [Edit](#)

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording [View >](#)

Download recording in .EDF format [Download](#)

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
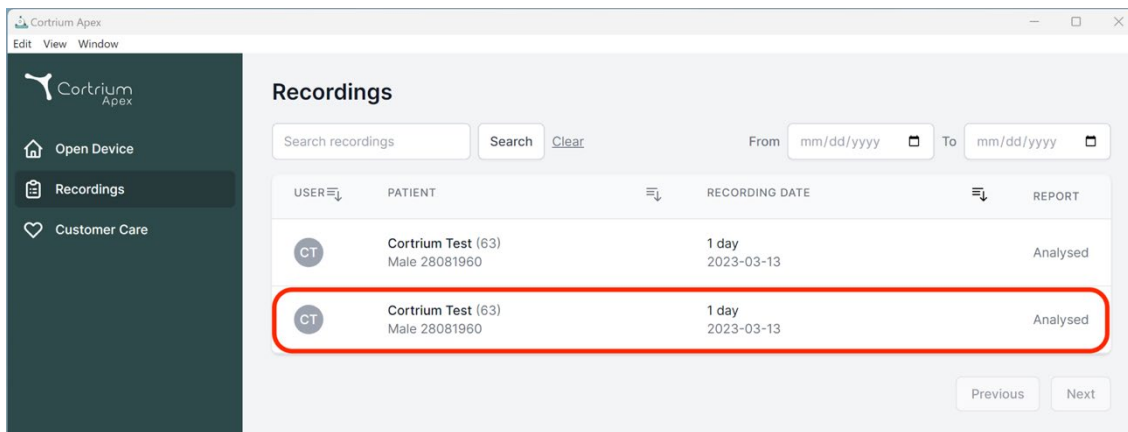
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

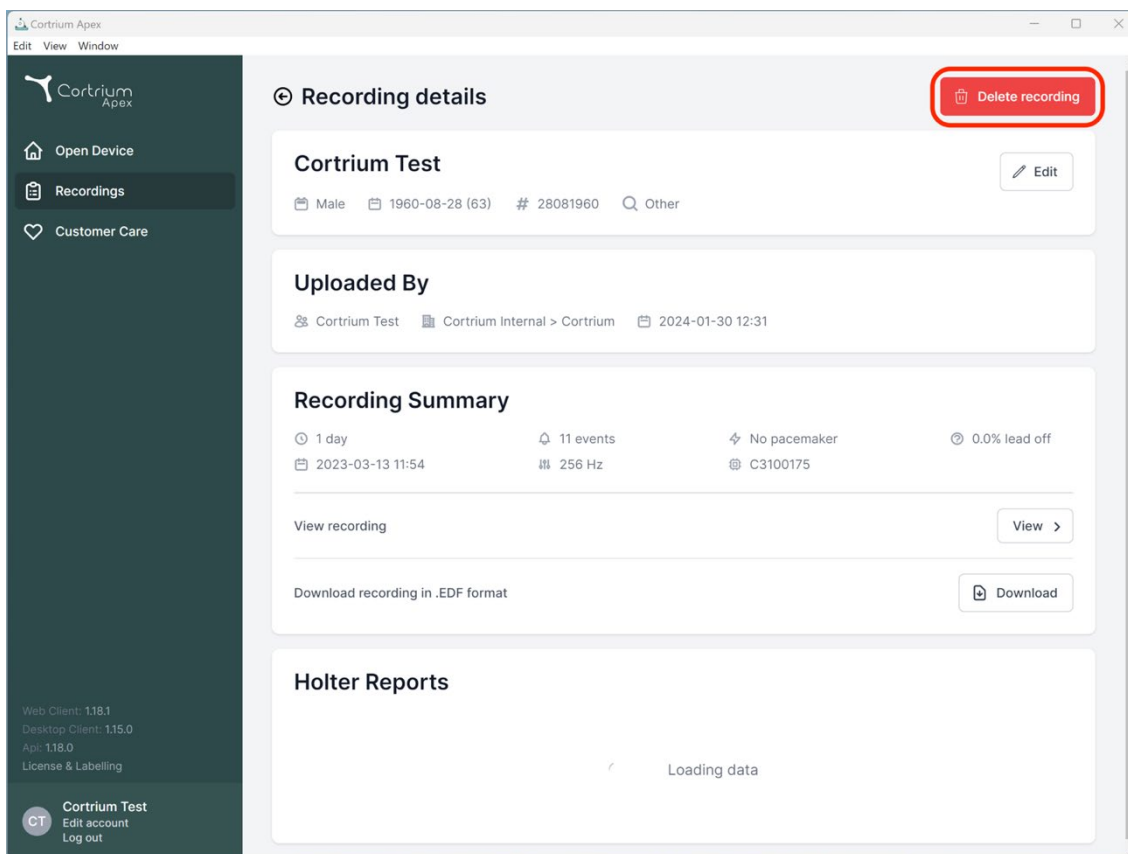Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Loading data

Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

 Customer Care

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

 Cortrium Test
Edit account
Log out

Recording details

Cortrium Test 

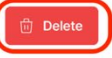
 Male  1960-08-28 (63)  # 28081960  Other

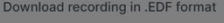
Uploaded By

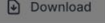
 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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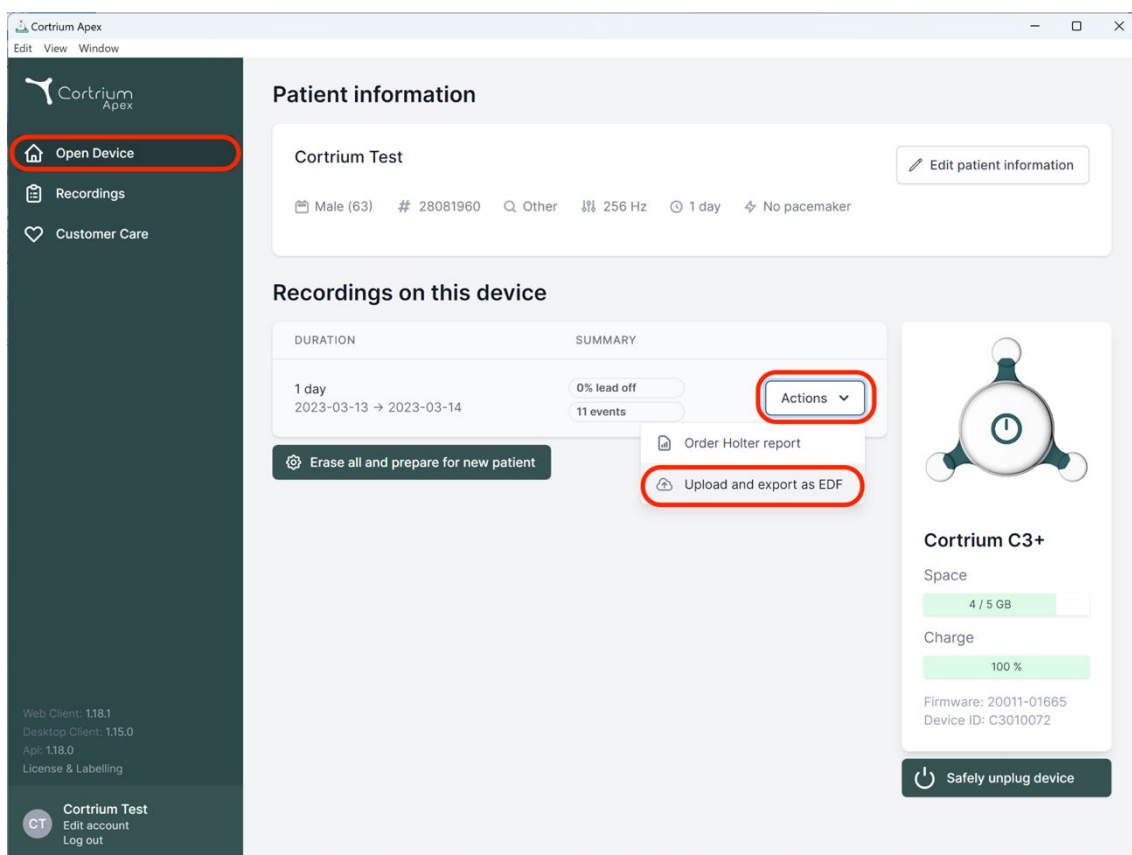
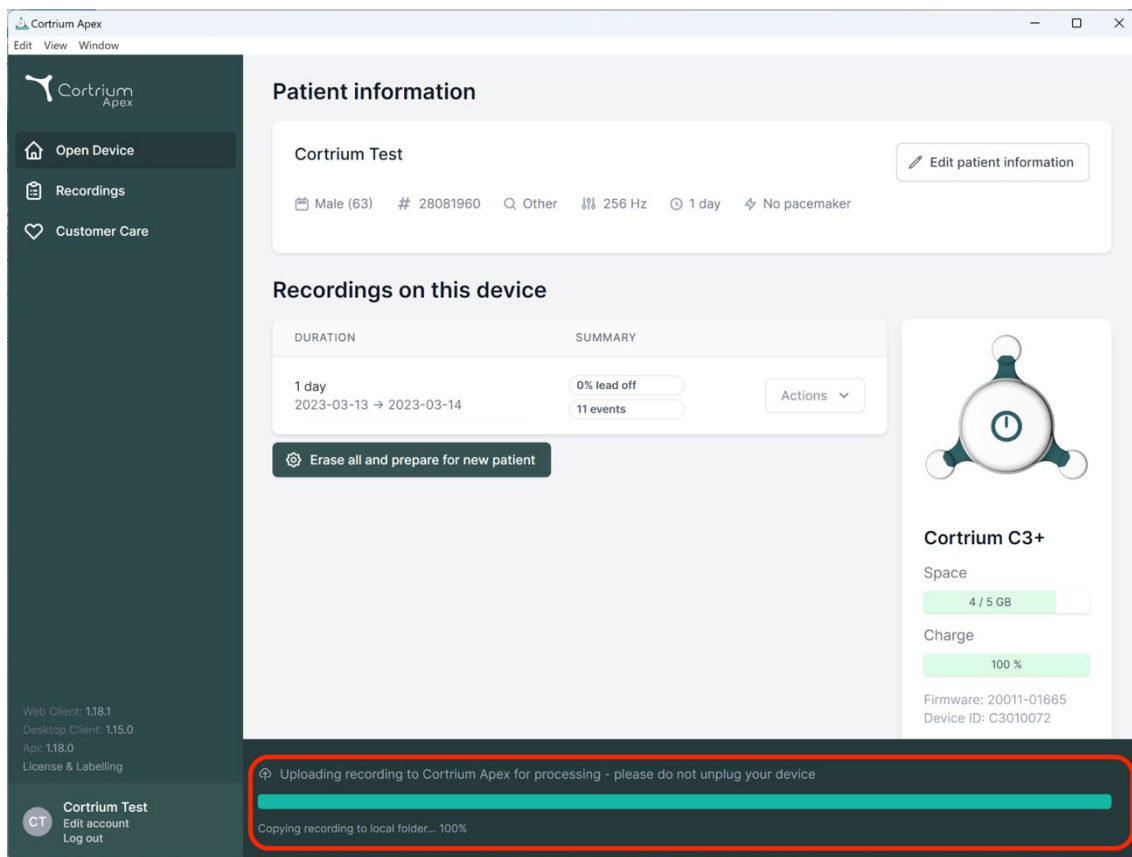
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

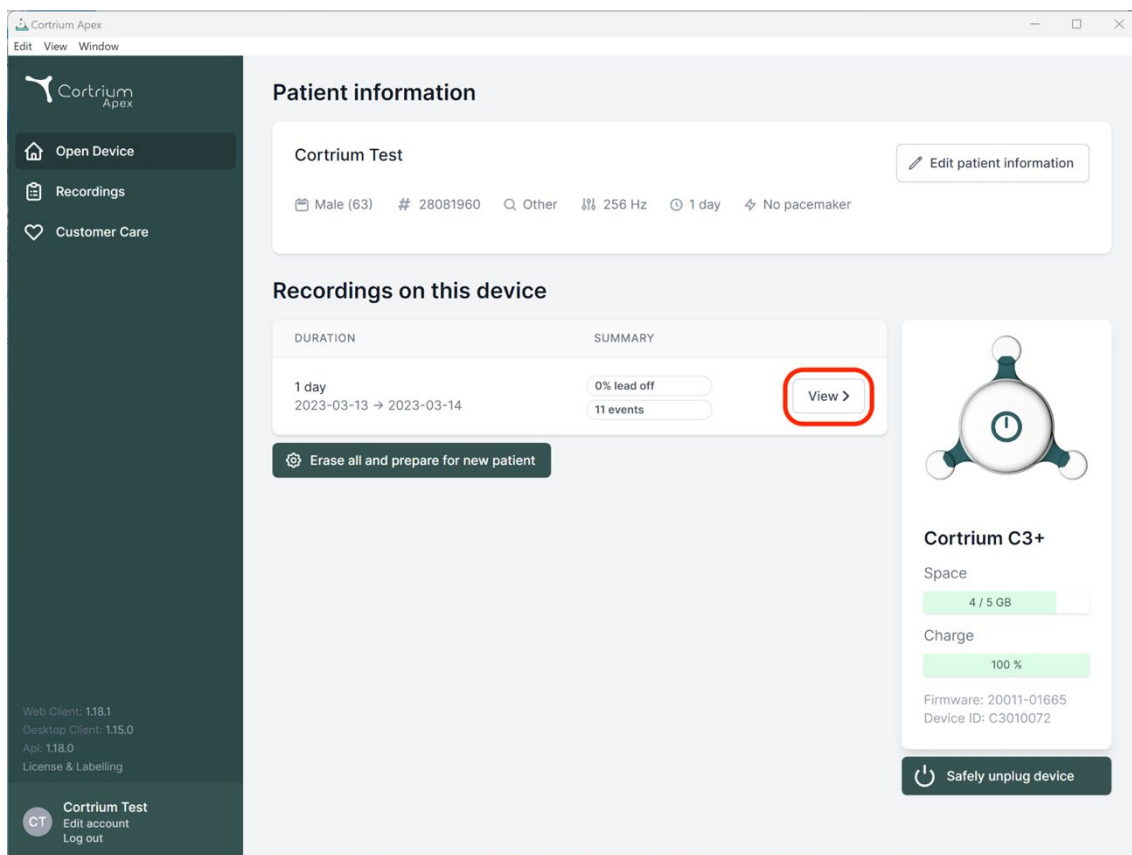
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
CT Edit account Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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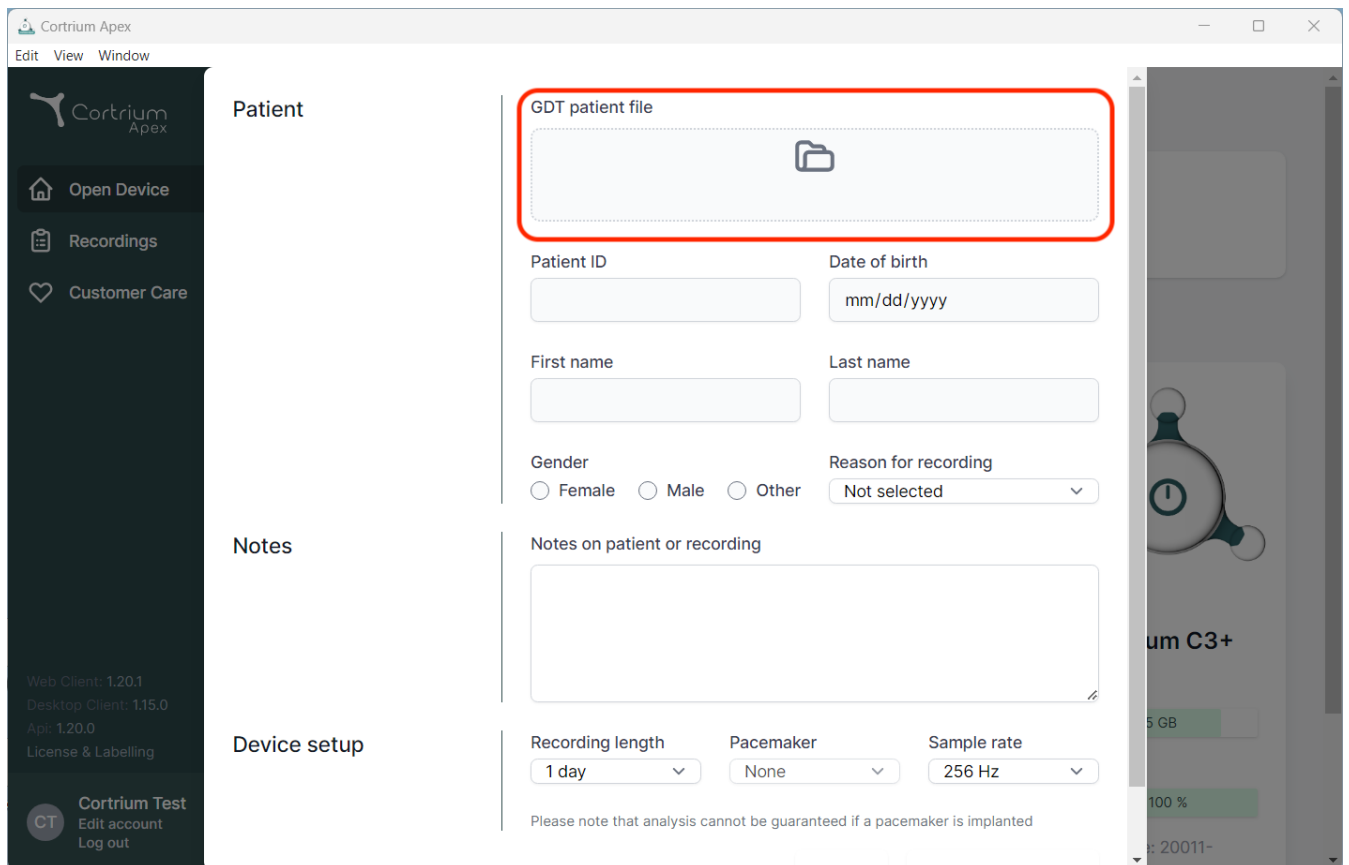
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", and "Api: 1.20.0". At the bottom of the sidebar is a user profile section for "Cortrium Test" with options to "Edit account" and "Log out".

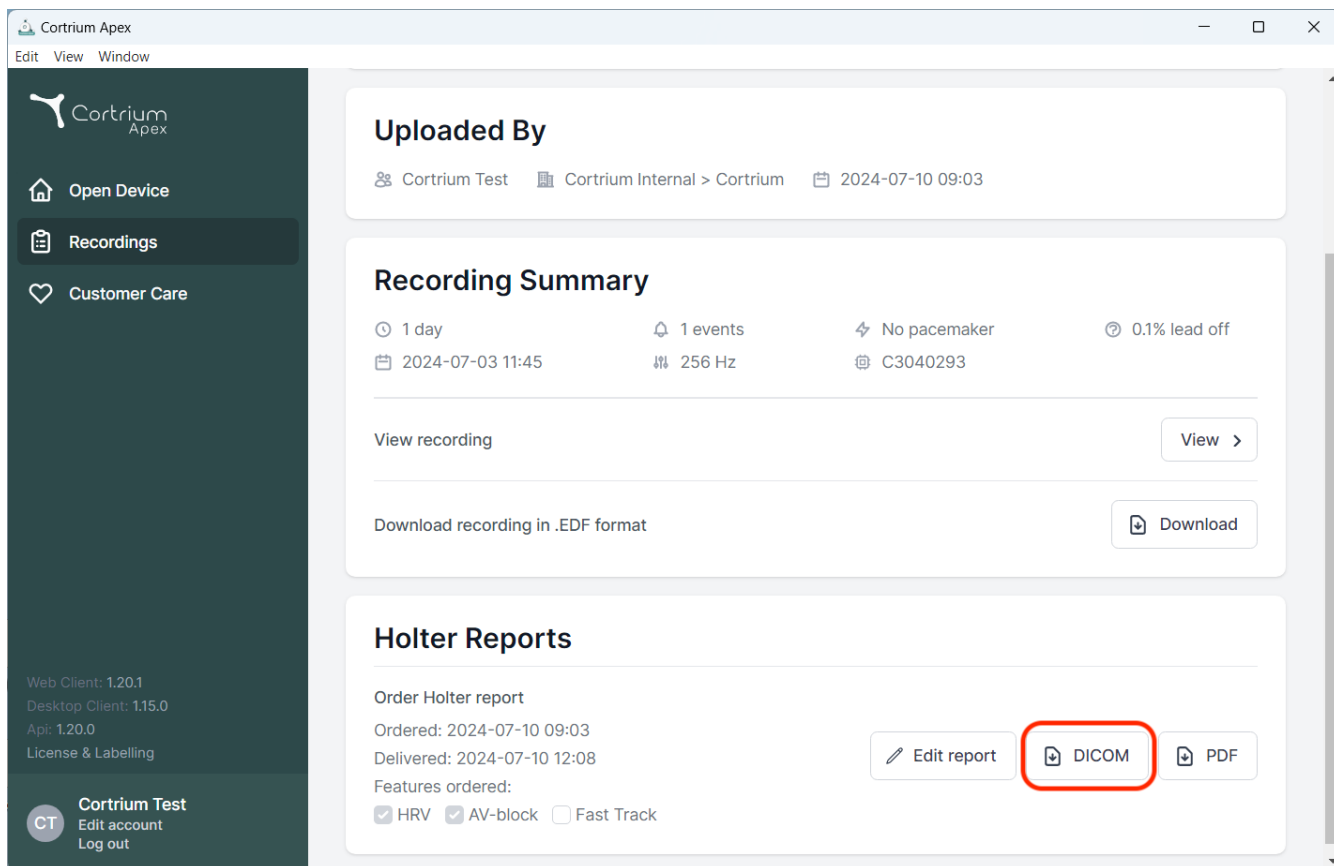
The main content area is divided into three sections: "Patient", "Notes", and "Device setup".

- Patient section:** Contains a "GDT patient file" upload area (highlighted with a red box) with a folder icon. Below this are input fields for "Patient ID", "Date of birth" (format mm/dd/yyyy), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown for "Reason for recording" (currently "Not selected").
- Notes section:** Labeled "Notes on patient or recording", it features a large text area for notes.
- Device setup section:** Includes dropdowns for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device model, "um C3+", is visible, showing a battery level of "100 %" and a storage indicator of "5 GB".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

The bottom left of the sidebar shows version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a user profile for 'Cortrium Test' with 'Edit account' and 'Log out' options.

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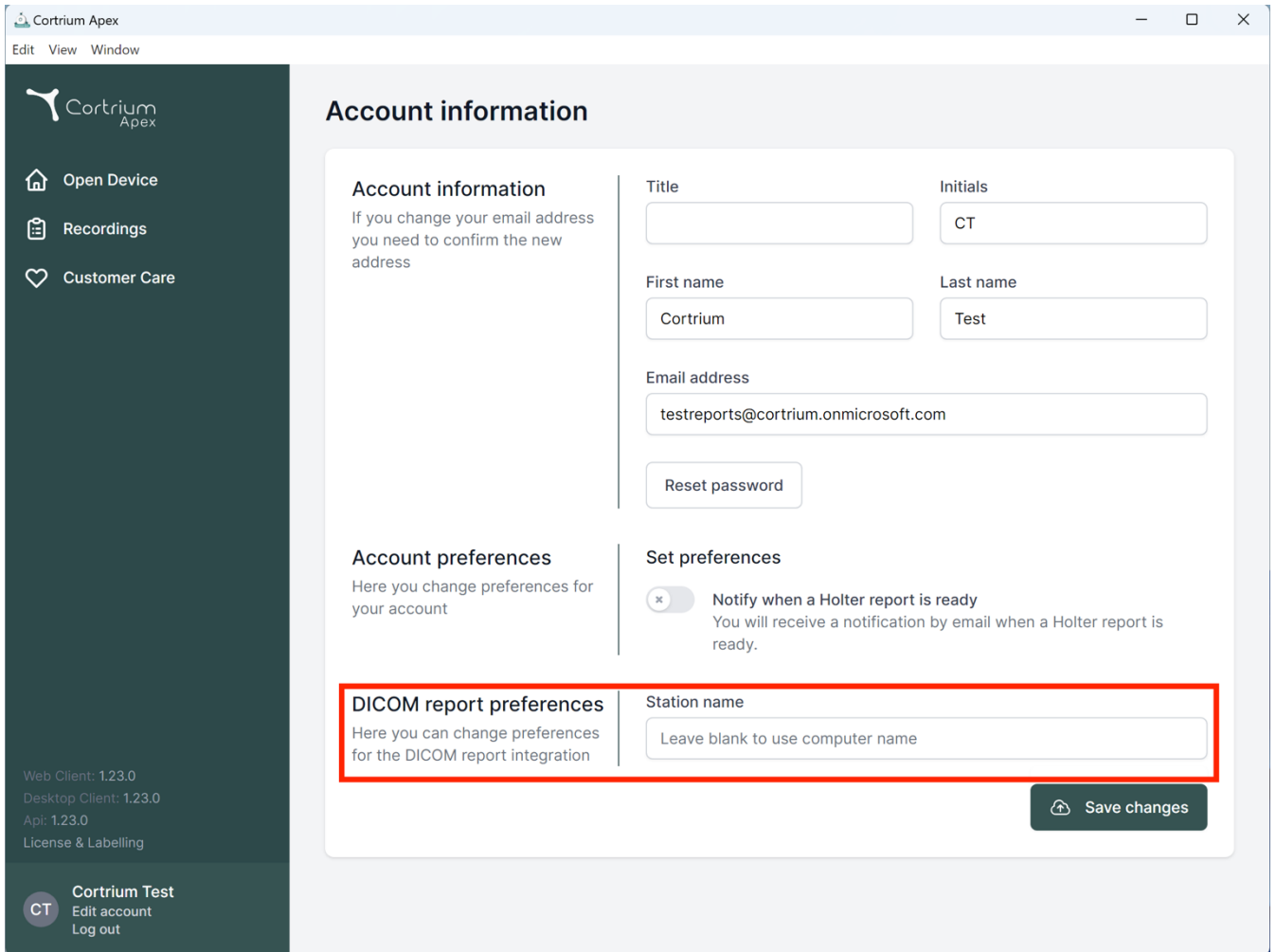Cortrium Apex
Edit View Window

Account information
If you change your email address you need to confirm the new address

Account preferences
Here you change preferences for your account

Set preferences
☐ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name
Leave blank to use computer name

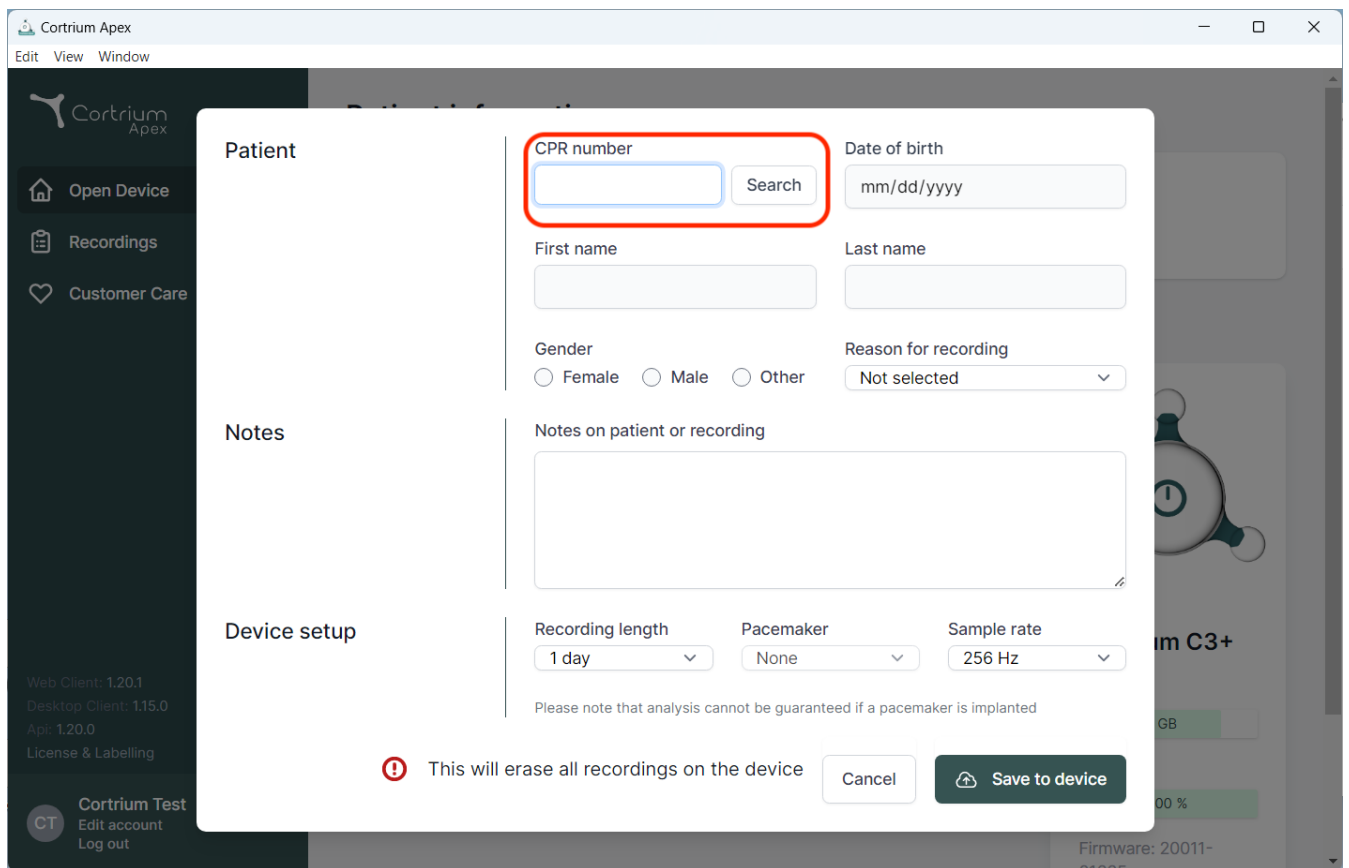
Save changes

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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Pacemaker

Sample rate

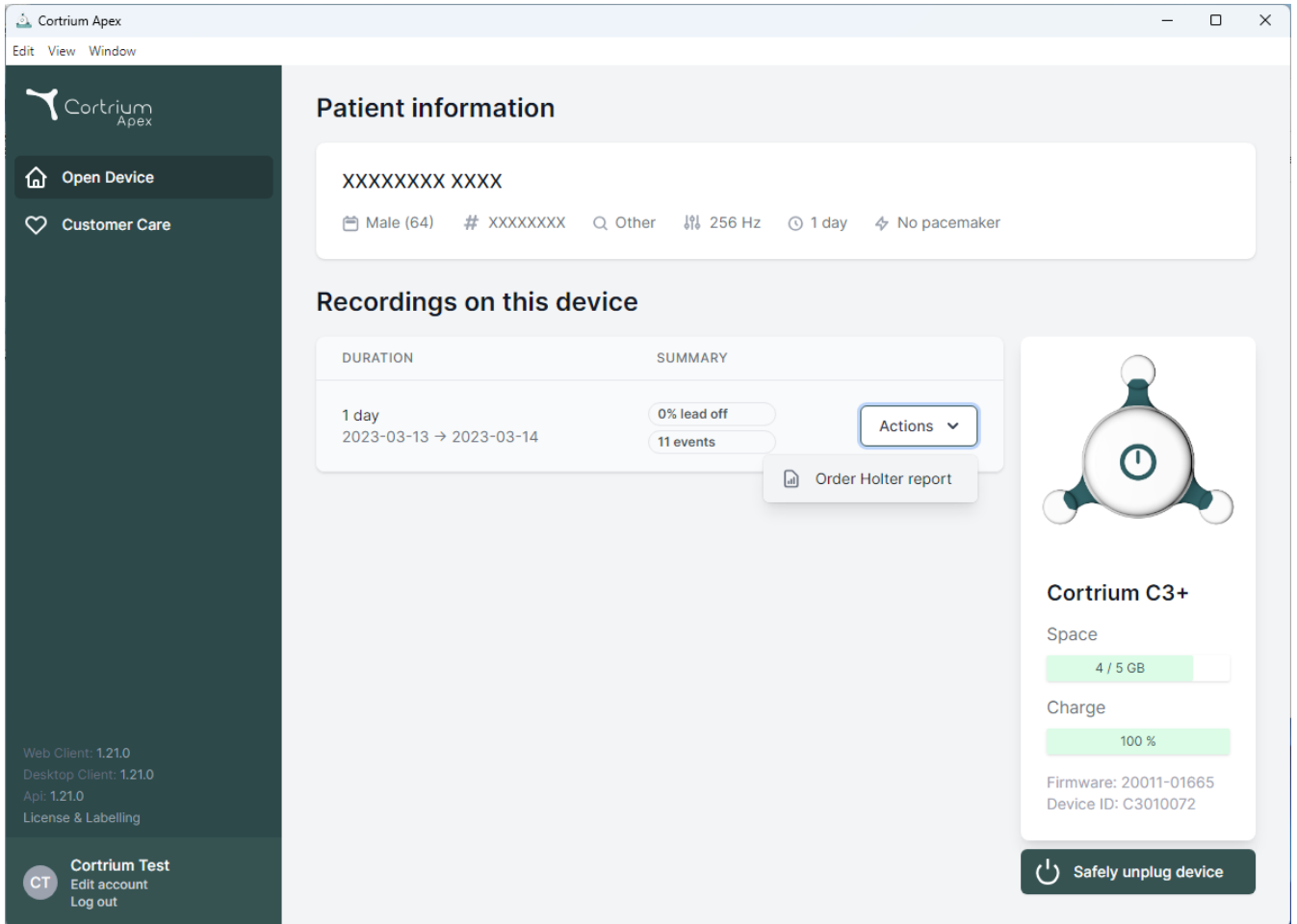
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' options.

The main content area is titled 'Patient information' and displays a patient ID 'XXXXXXXX XXXX'. Below this, patient details are shown: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with two columns: 'DURATION' and 'SUMMARY'. The table contains one entry for a '1 day' recording from '2023-03-13 → 2023-03-14'. The summary for this recording shows '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the summary, with an option to 'Order Holter report'. On the right side of the interface, there is a section for the 'Cortrium C3+' device. It includes a diagram of the device, its name, and status indicators for 'Space' (4 / 5 GB) and 'Charge' (100 %). Below these, the 'Firmware' (20011-01665) and 'Device ID' (C3010072) are listed. At the bottom right, there is a 'Safely unplug device' button with a power icon.

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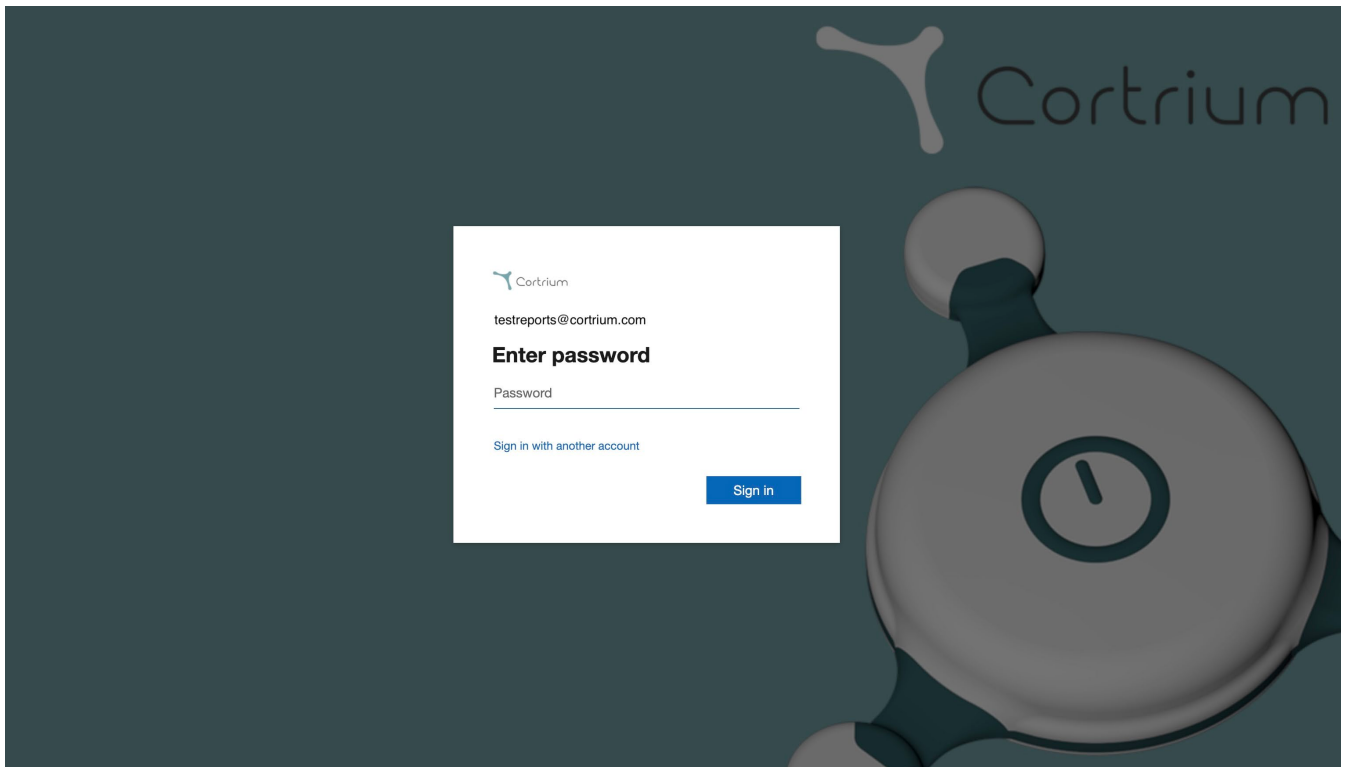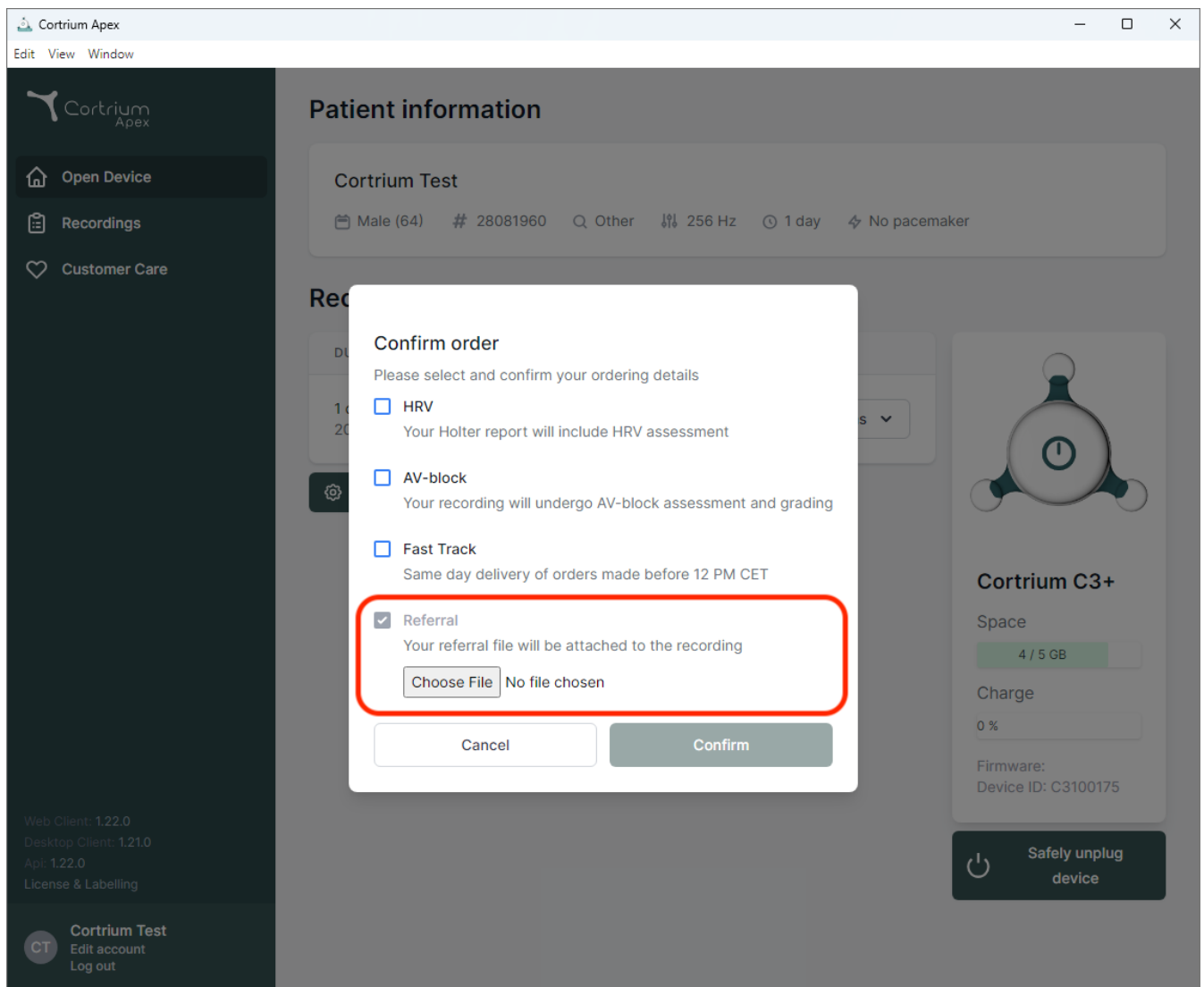
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
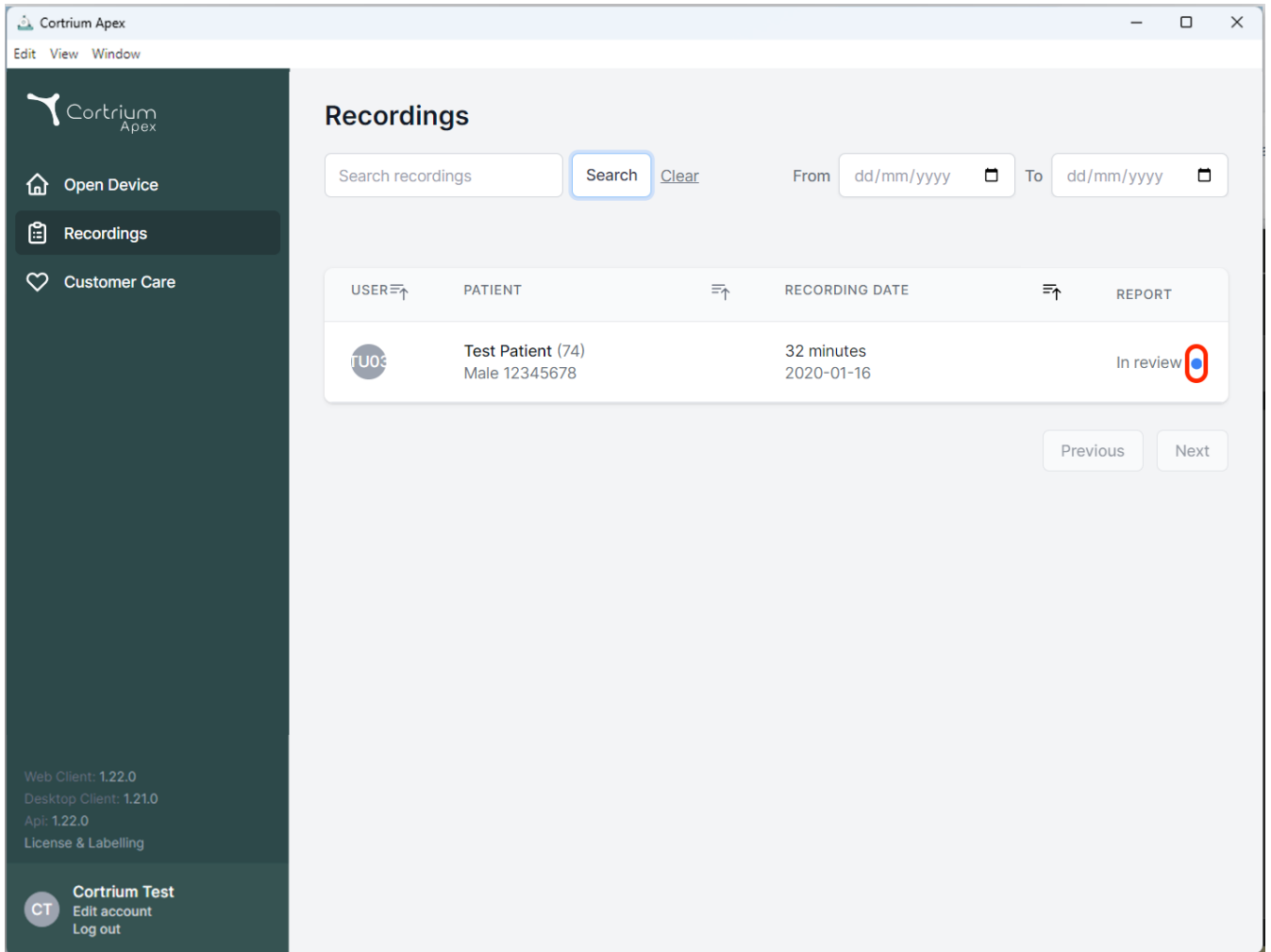
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". The main area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. A single recording is listed for user TU03, patient Test Patient (74), with a duration of 32 minutes and a date of 2020-01-16. The status "In review" is shown next to a blue dot icon, which is circled in red. At the bottom of the sidebar, there is a "Cortrium Test" section with links for "Edit account" and "Log out".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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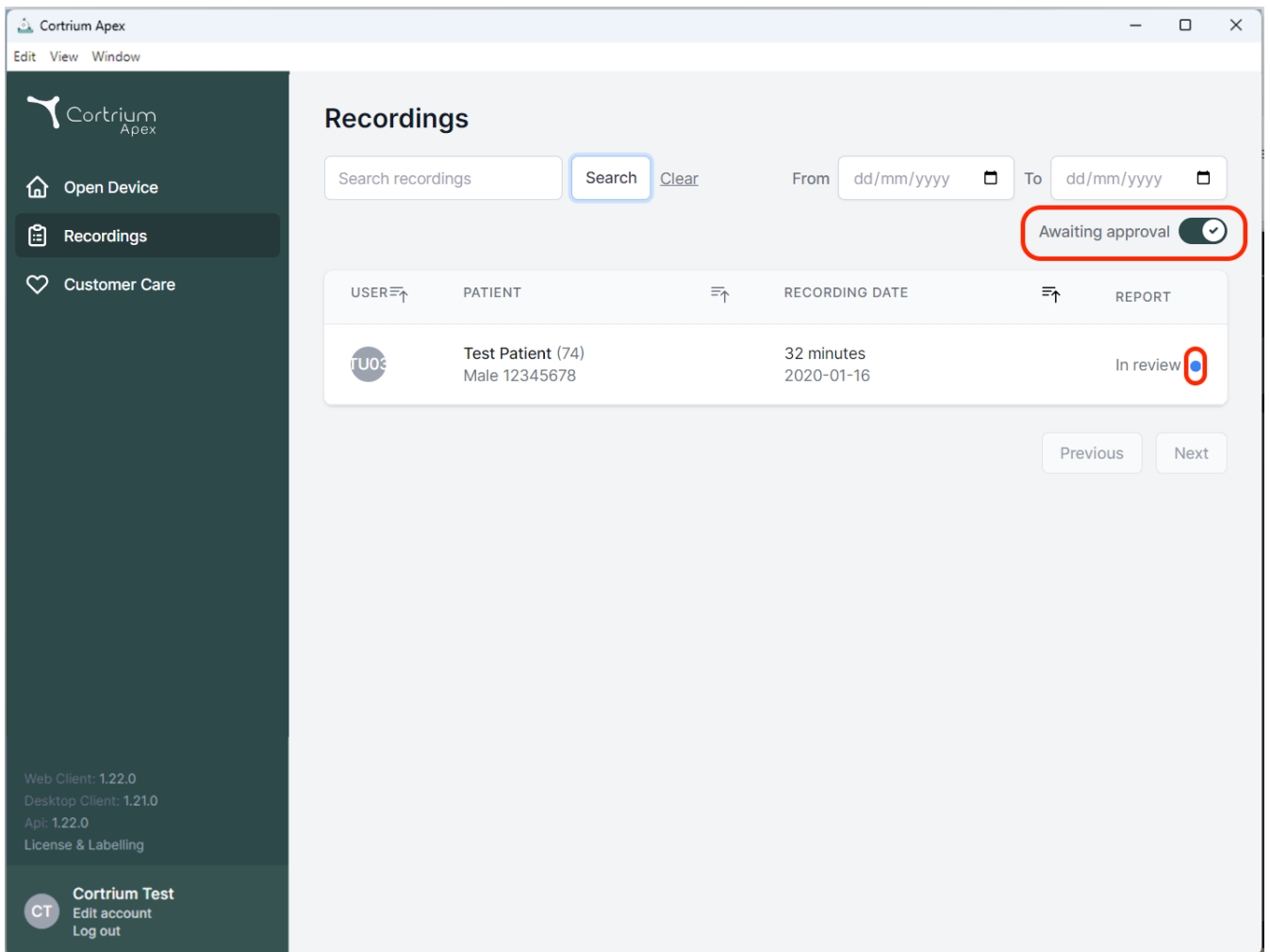
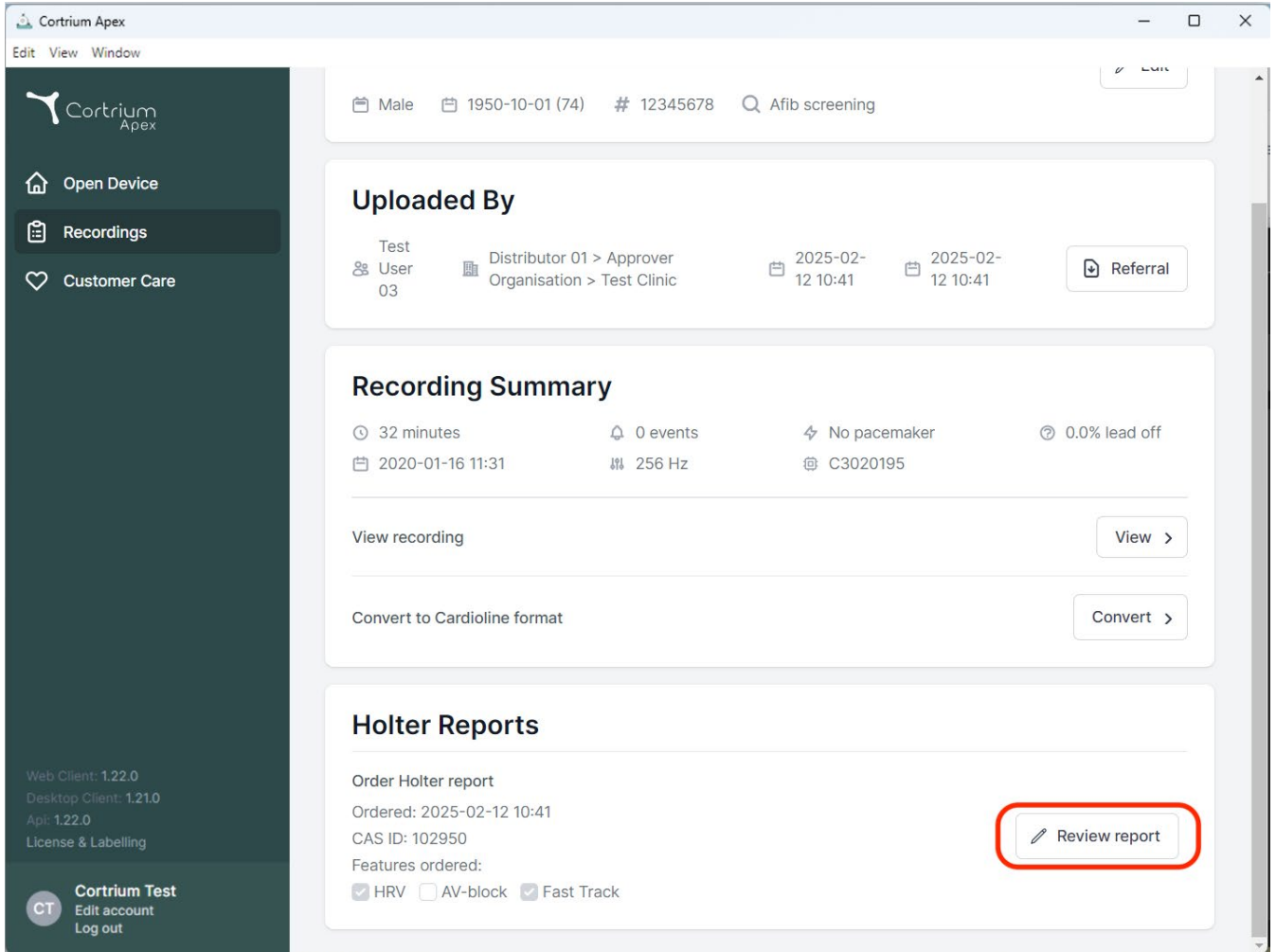
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and three main menu items: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into several sections:

- Header Information:** Patient details including 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'.
- Uploaded By:** A section showing the test user 'User 03', the distributor 'Distributor 01 > Approver Organisation > Test Clinic', and two timestamps: '2025-02-12 10:41' and '2025-02-12 10:41'. A 'Referral' button is also present.
- Recording Summary:** A section providing key recording metrics: '32 minutes', '0 events', 'No pacemaker', and '0.0% lead off'. It also includes the recording date '2020-01-16 11:31', a frequency of '256 Hz', and a device ID 'C3020195'. Below this, there are buttons for 'View recording' and 'Convert to Cardioline format'.
- Holter Reports:** A section for managing holter reports. It includes an 'Order Holter report' button, the order date 'Ordered: 2025-02-12 10:41', and the CAS ID 'CAS ID: 102950'. Under 'Features ordered', there are checkboxes for 'HRV' (checked), 'AV-block' (unchecked), and 'Fast Track' (checked). A 'Review report' button is highlighted with a red rectangle.

The bottom left corner of the interface shows version information: 'Web Client: 1.22.0', 'Desktop Client: 1.21.0', 'Api: 1.22.0', and 'License & Labelling'. The bottom right corner features a 'Cortrium Test' section with a 'CT' icon, 'Edit account', and 'Log out' options.

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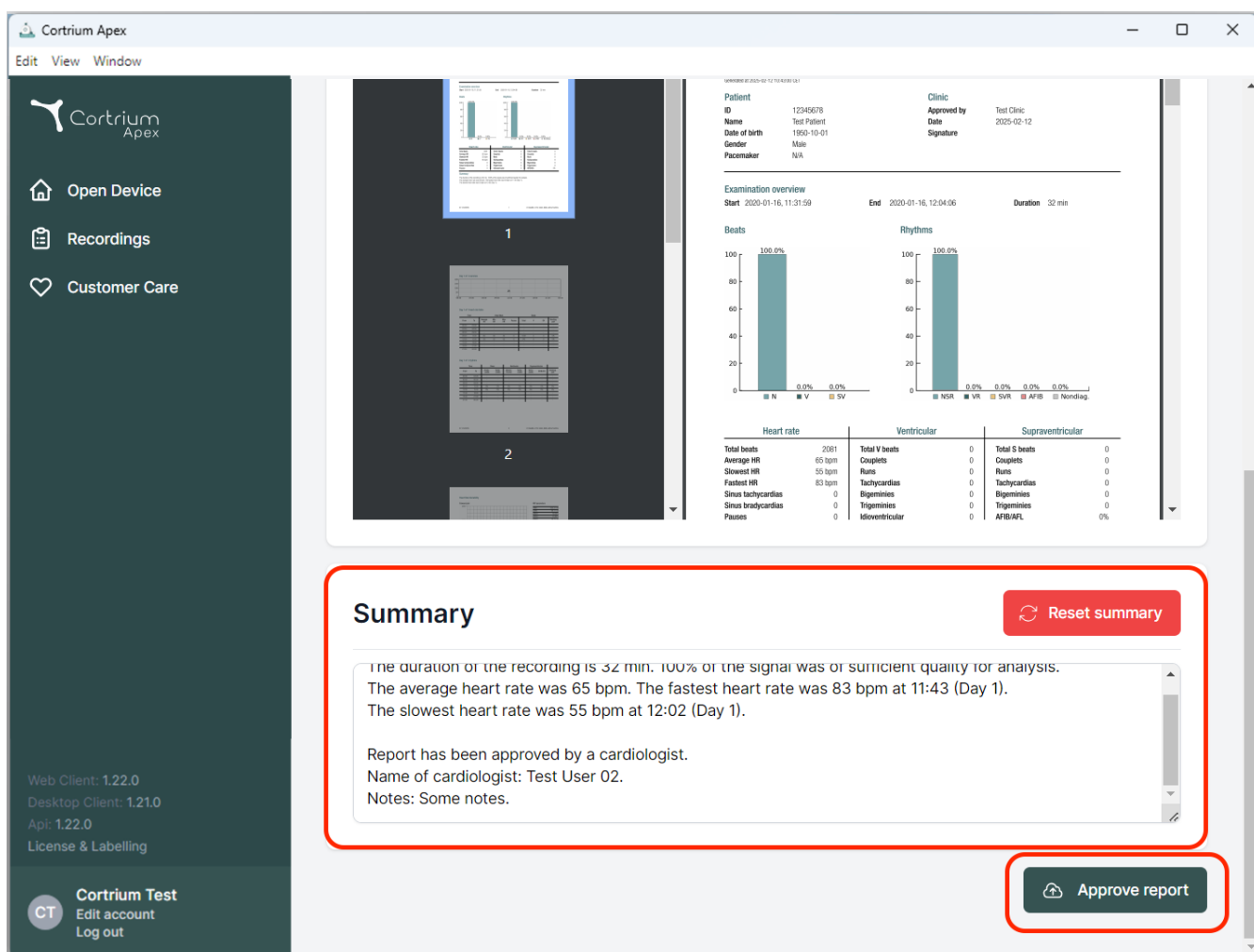Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
App: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient
ID: 12345678
Name: Test Patient
Date of birth: 1990-10-01
Gender: Male
Pacemaker: N/A

Clinic
Approved by: Test Clinic
Date: 2025-02-12
Signature:

Examination overview
Start: 2020-01-16, 11:31:59
End: 2020-01-16, 12:04:06
Duration: 32 min

Beats
100.0%
0.0%
0.0%

Rhythms
100.0%
0.0%
0.0%
0.0%
0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardias	0	Bigeminy	0	Bigeminy	0
Sinus bradycardias	0	Trigeminy	0	Trigeminy	0
Pauses	0	Midventricular	0	AF/IB/AF/IB	0%

Summary [Reset summary](#)

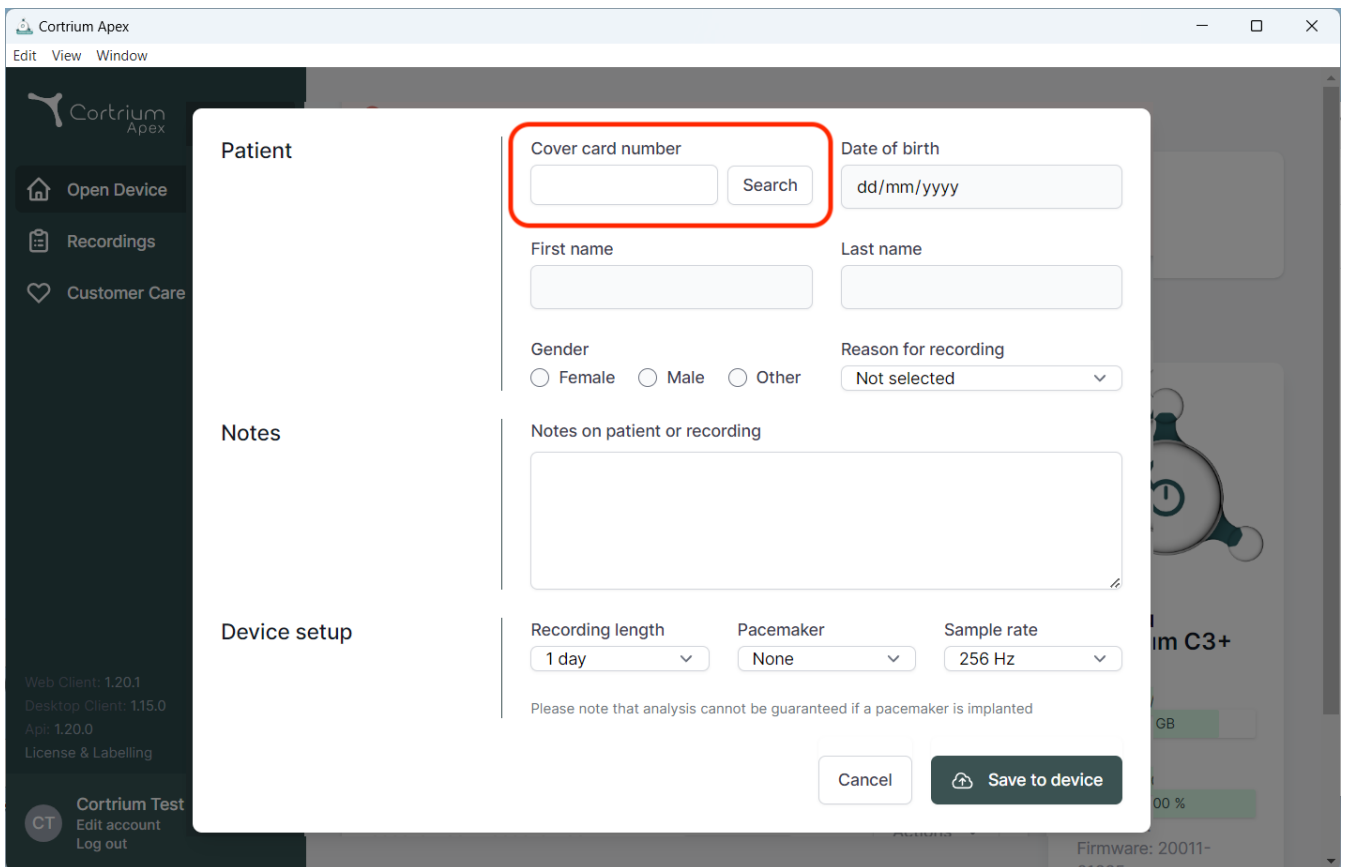
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot displays the Cortrium Apex software interface. A central dialog box is open, titled "Patient". The dialog box contains the following fields and options:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangle.
- Date of birth:** A text input field with the placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes on patient or recording:** A large text area for notes.
- Device setup:** A section containing:
 - Recording length:** A dropdown menu showing "1 day".
 - Pacemaker:** A dropdown menu showing "None".
 - Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom right of the dialog box, there are two buttons: "Cancel" and "Save to device".

The background interface shows a sidebar with "Open Device", "Recordings", and "Customer Care" options. The top bar includes "Edit", "View", and "Window" menus. The bottom status bar shows "Cortrium Test", "Edit account", "Log out", and "Firmware: 20011-01655".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Taani
E-post: info@cortrium.com • Veebisait: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47