Cortrium Apex 1.27

Cortrium Apex 1.27

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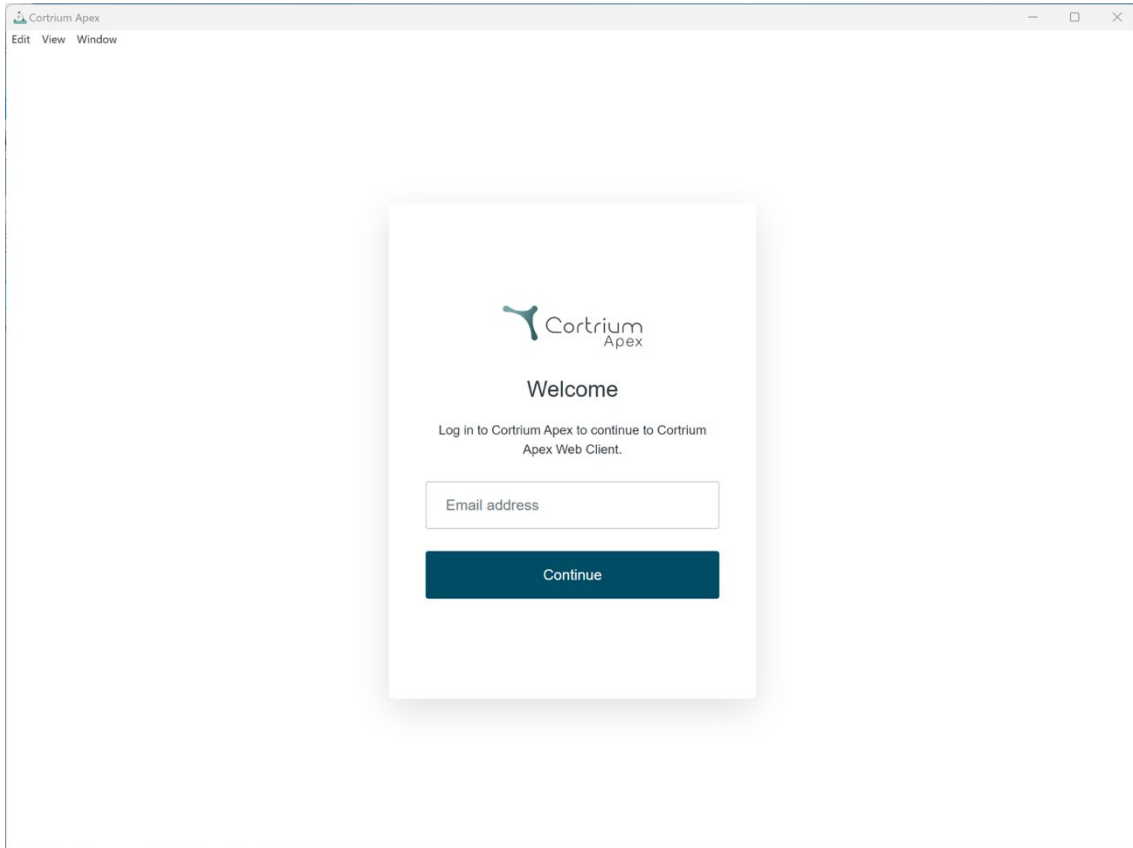
The screenshot shows a web browser window titled "Cortrium Apex". The browser's menu bar includes "Edit", "View", and "Window". The main content area displays a login form with the following elements:

- Cortrium Apex logo
- Section header: **Welcome**
- Text: "Log in to Cortrium Apex to continue to Cortrium Apex Web Client."
- Input field: "Email address"
- Button: "Continue"

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The screenshot shows the Cortrium Apex web application. The left sidebar contains navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Account information' and contains several sections:

- Account information:** Fields for Title, Initials, First name, Last name, and Email address. A 'Reset password' button is also present.
- Account preferences:** A section for changing account preferences.
- DICOM report preferences:** A section for changing preferences for DICOM report integration.
- Local computer preferences:** A section for changing preferences for the local computer. It includes a checkbox labeled 'Keep users signed in on this computer' (labeled with a red '2') and a description: 'The last user will be automatically signed in again when starting Apex.'

A 'Save changes' button is located at the bottom right of the 'Local computer preferences' section. In the bottom left corner, a user profile for 'Cortrium Test' is shown (labeled with a red '1'), with options to 'Edit account' and 'Log out'.

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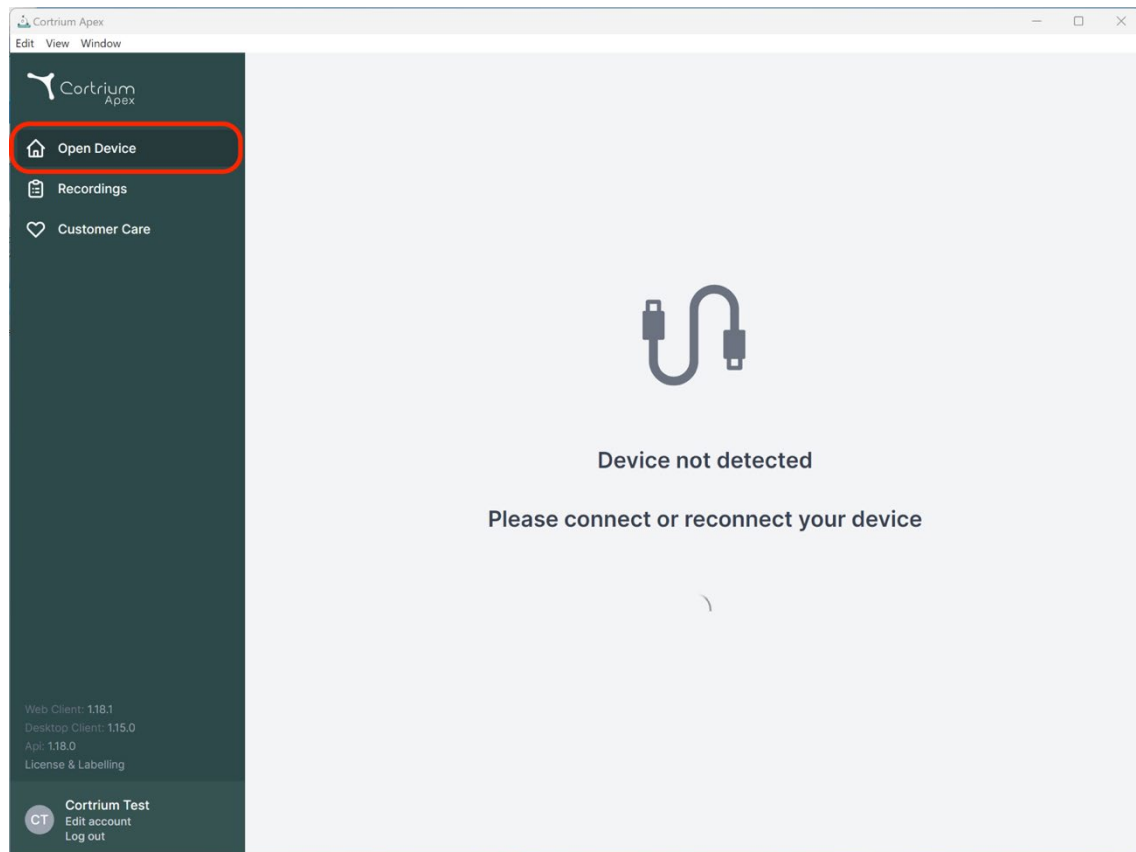
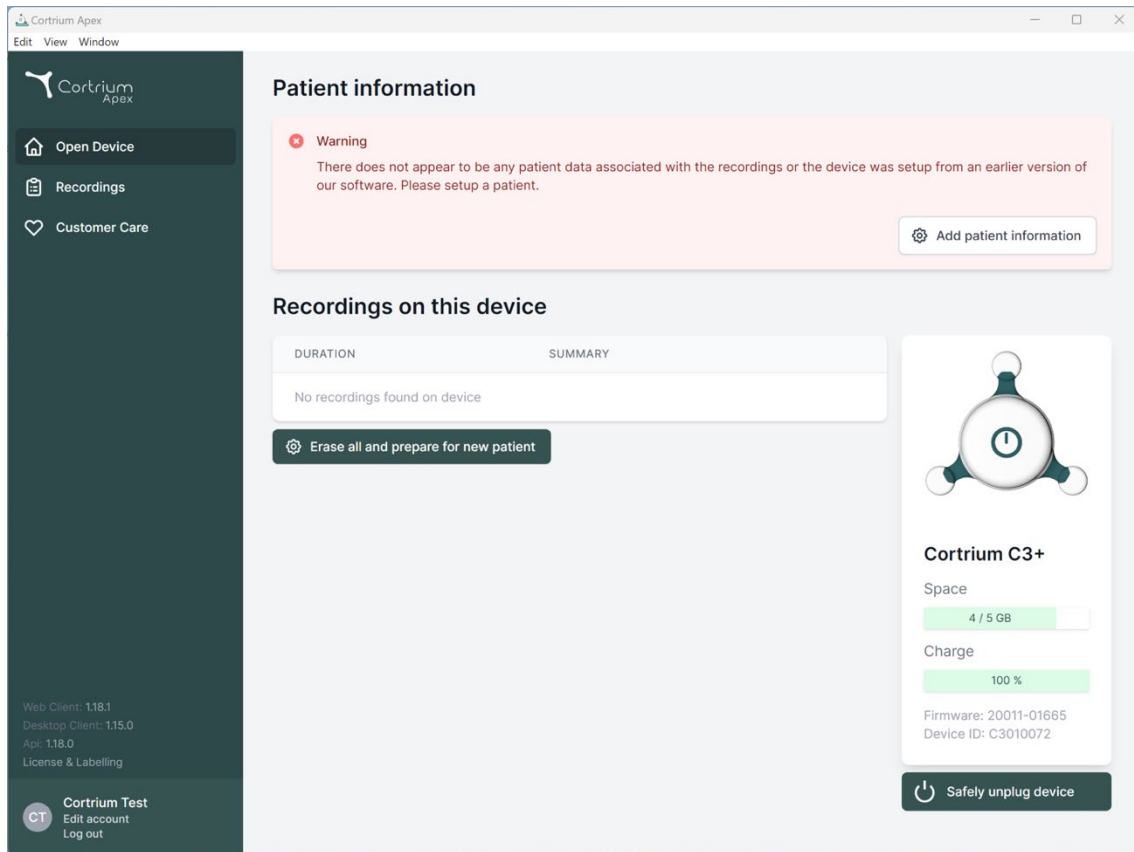
Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Save to device

Warning This will erase all recordings on the device

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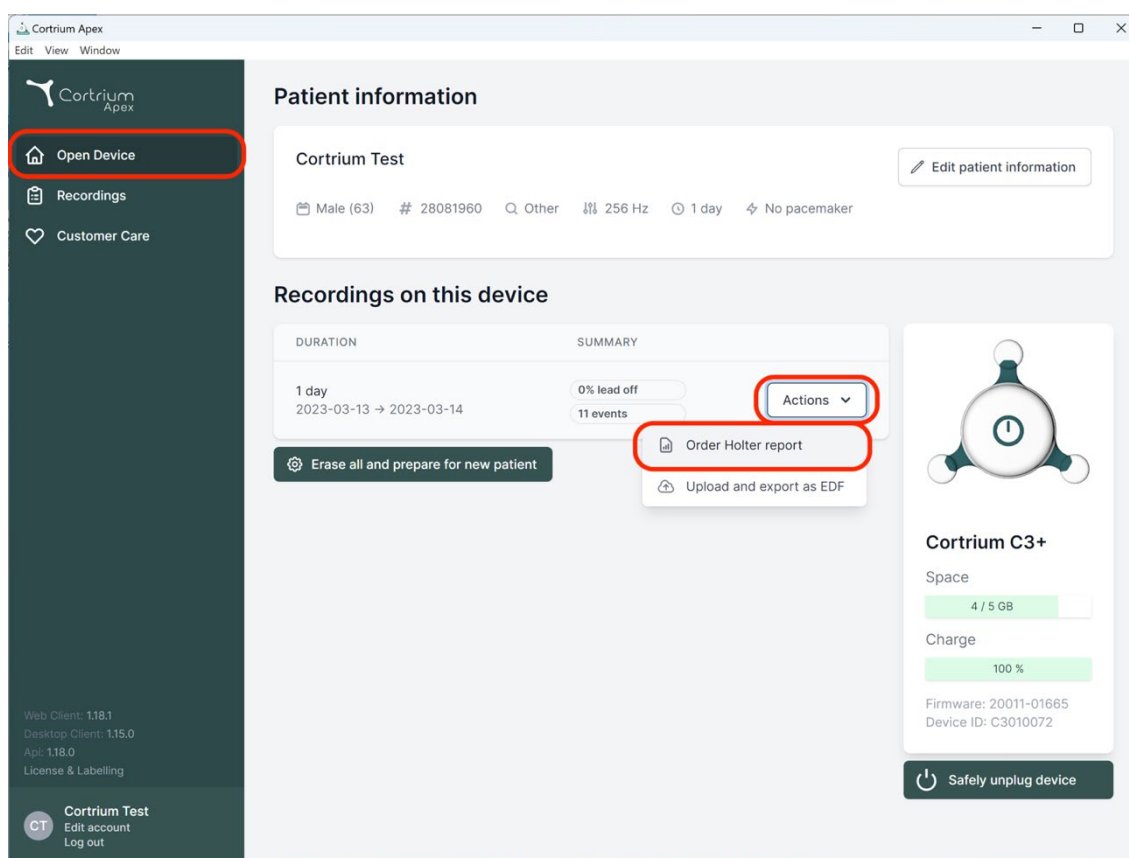
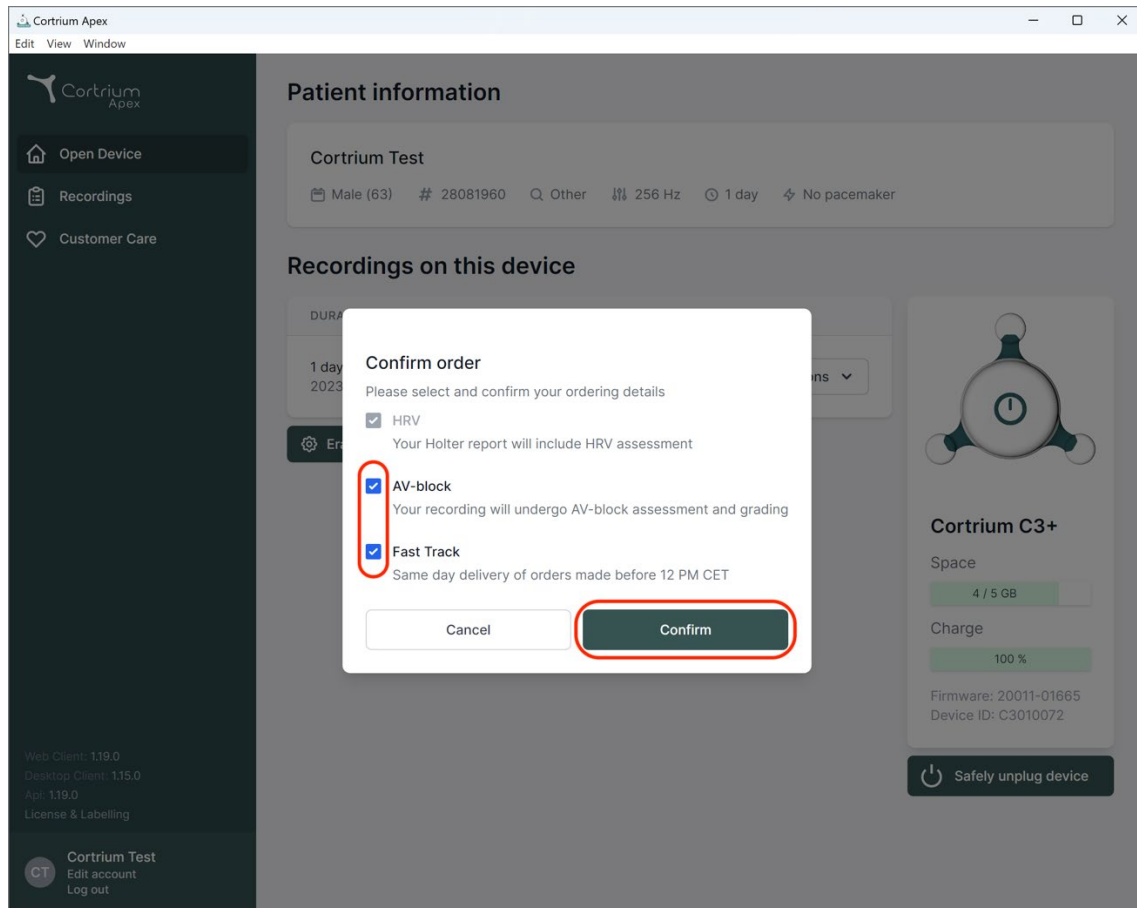
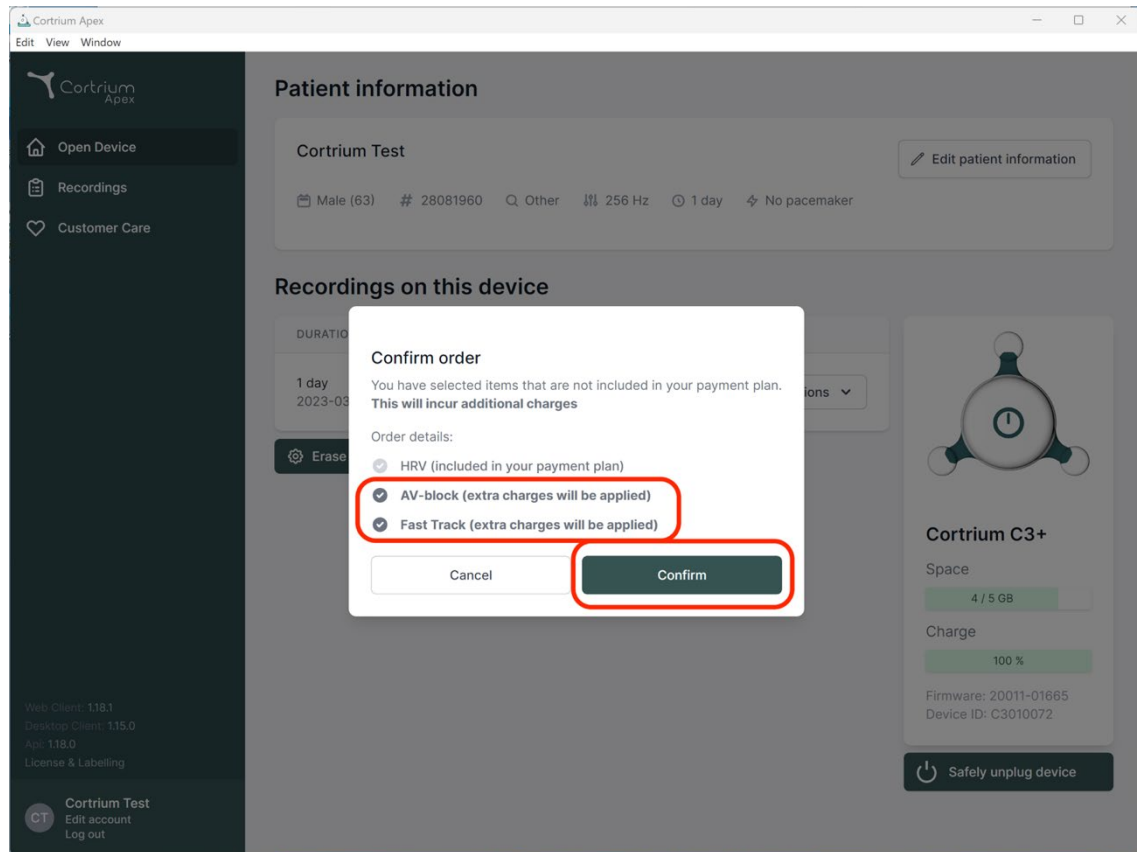
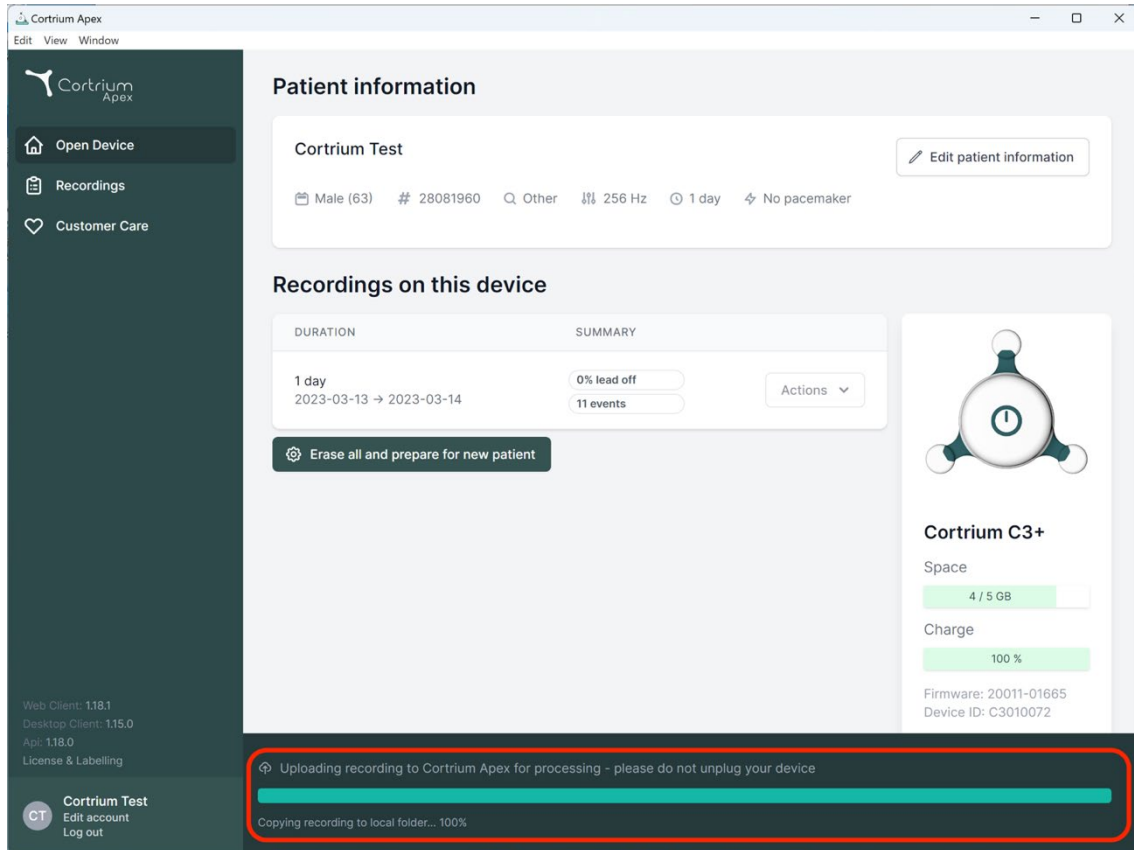
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and test duration (1 day). Below this, the 'Recordings on this device' section shows a table with columns for DURATION and SUMMARY. The table lists a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A button 'Erase all and prepare for new patient' is visible. To the right, a device status card for 'Cortrium C3+' shows 4/5 GB space, 100% charge, and firmware details. At the bottom, a red-bordered box highlights a message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' with a progress bar at 100% and the text 'Copying recording to local folder... 100%'.

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
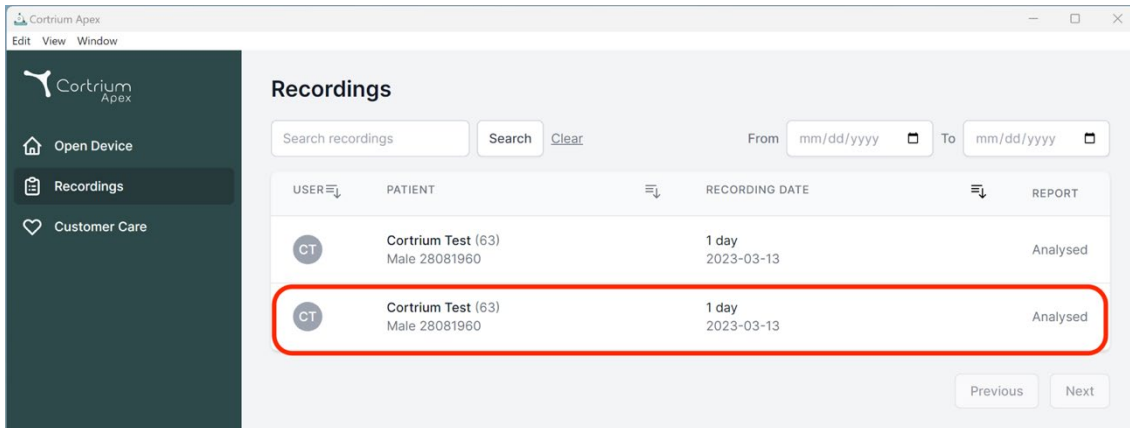
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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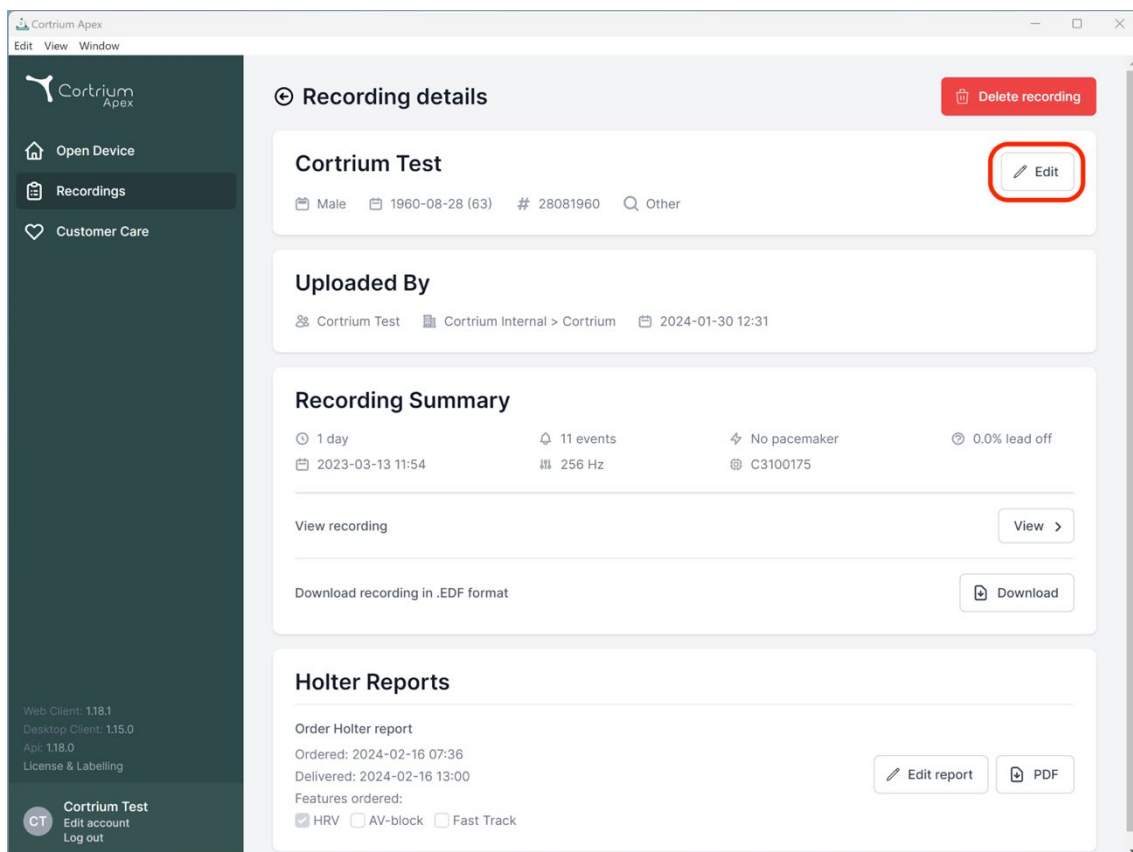2023-03-13 11:54 256 Hz C3100175

View recording View

Download recording in .EDF format Download

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
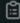
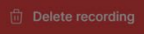
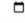
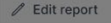
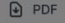
Order Holter report


Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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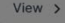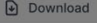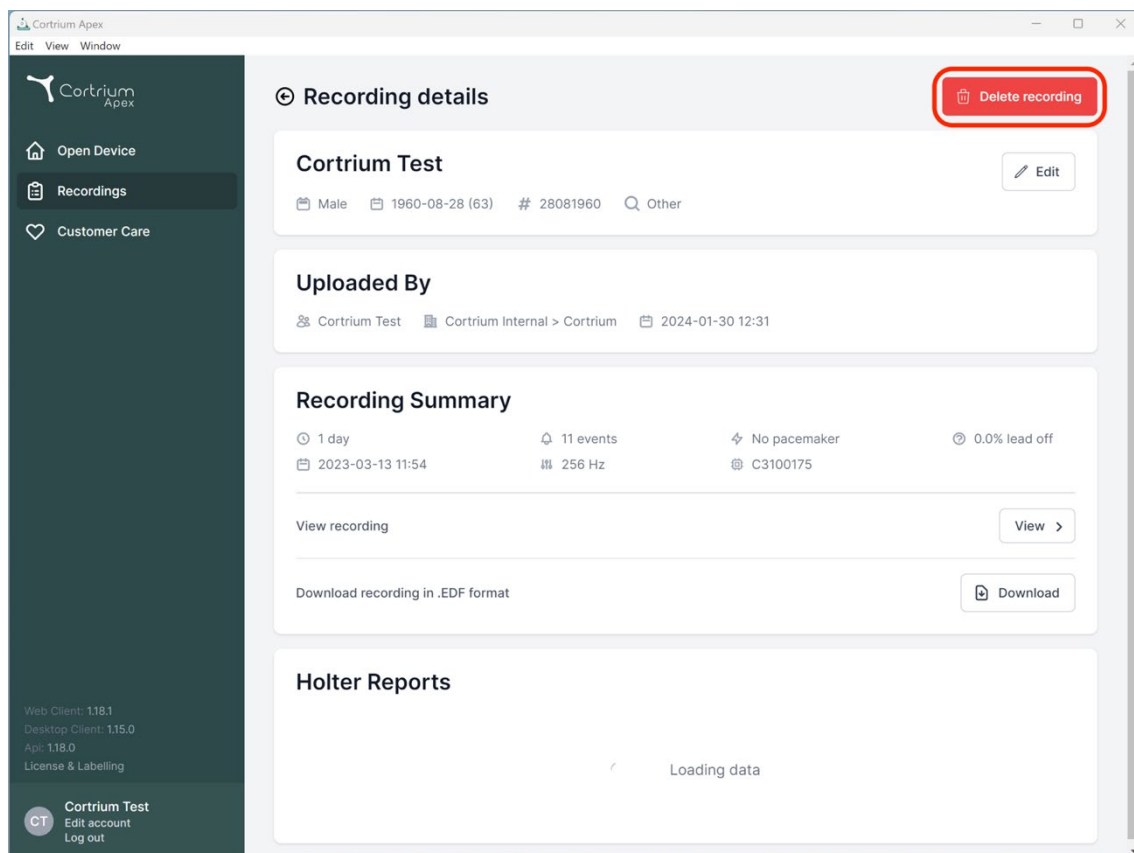
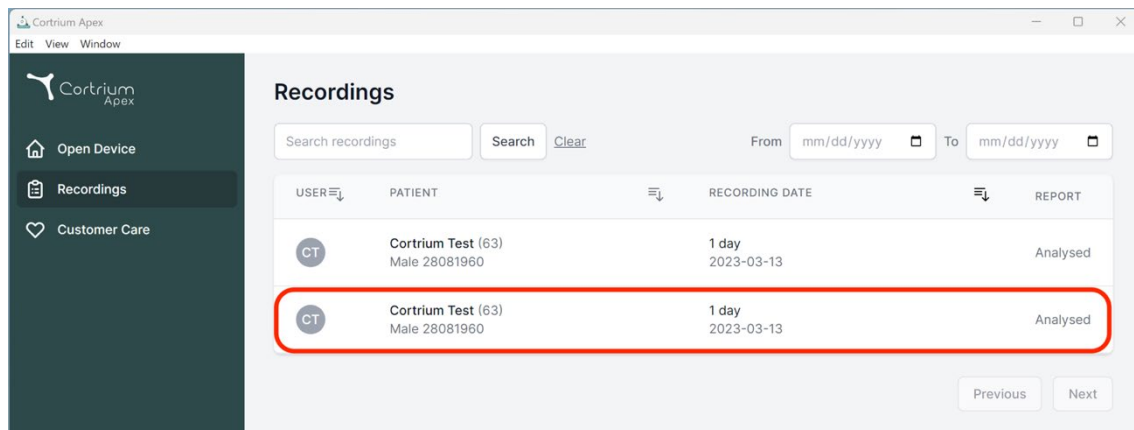
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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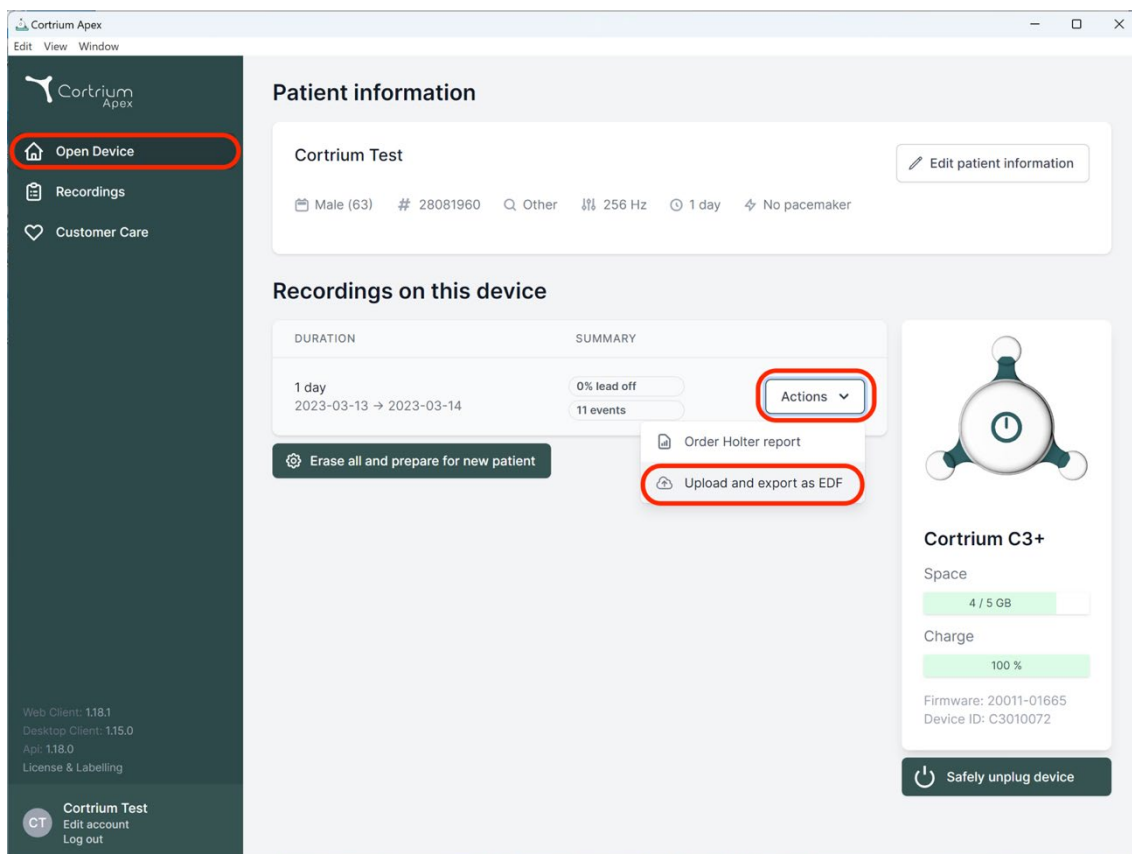
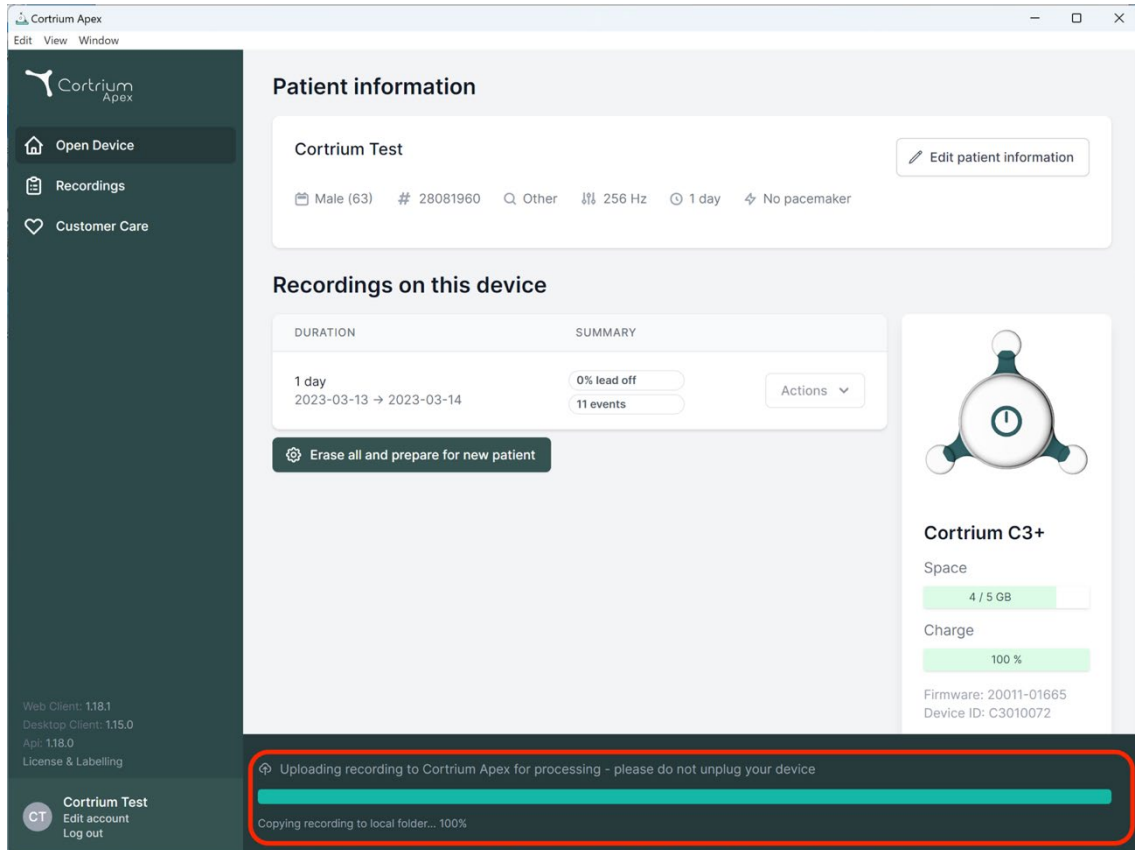
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

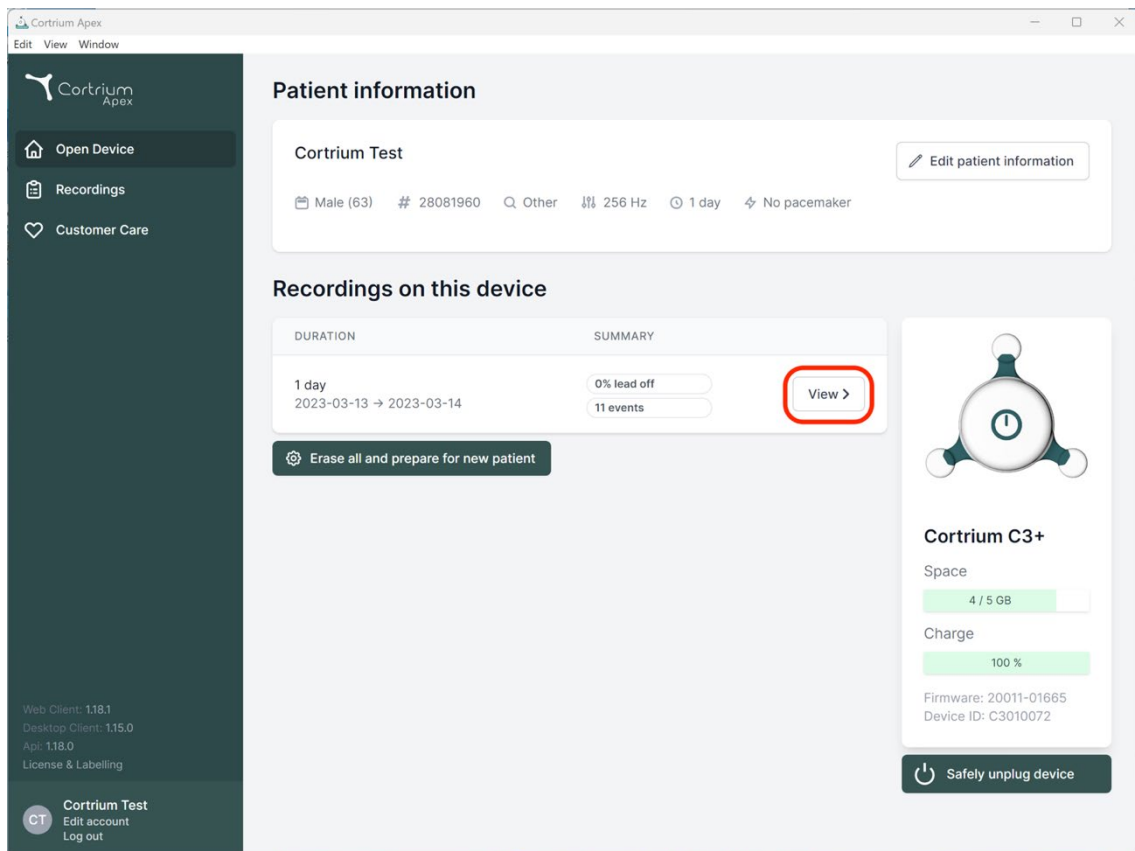
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
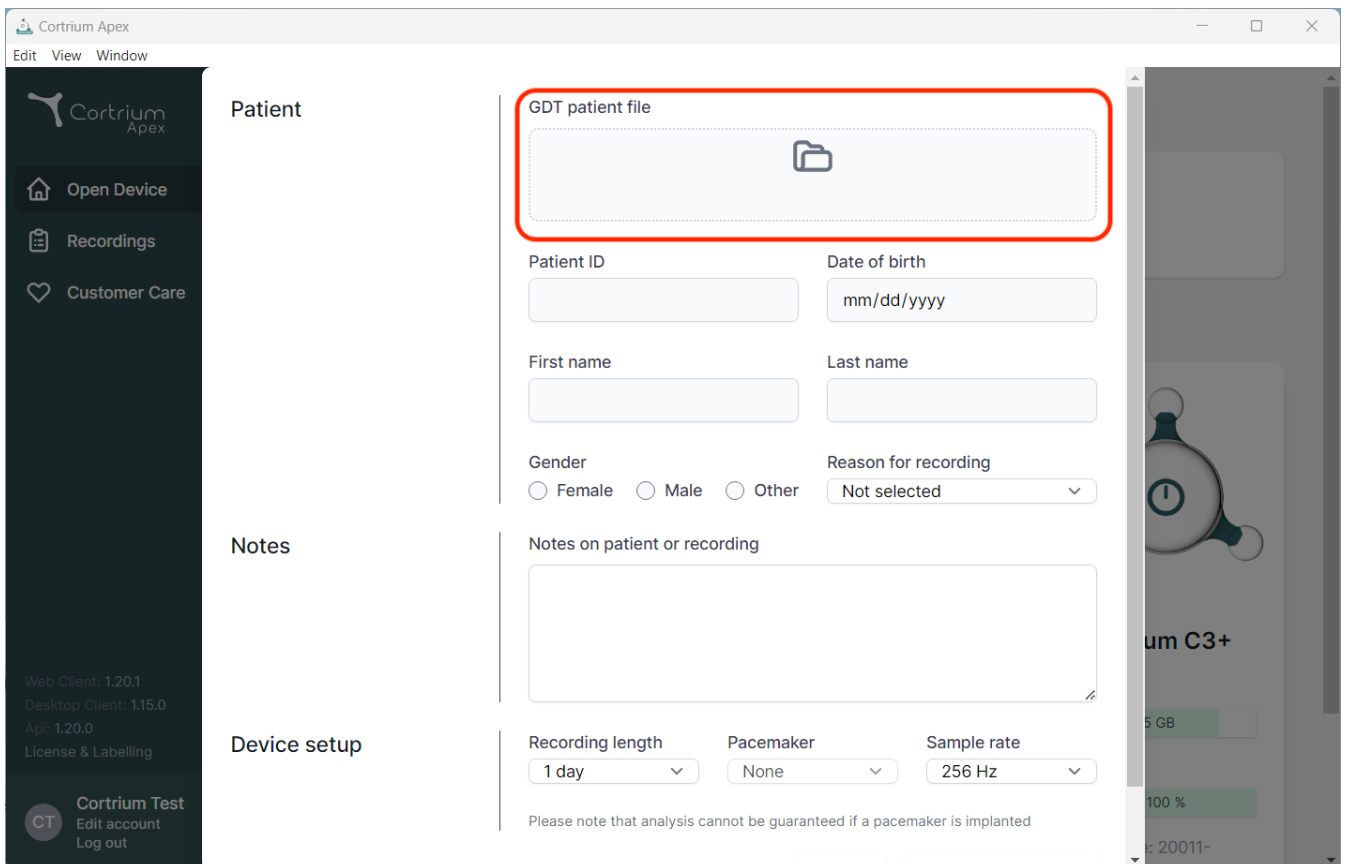
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details for Web Client (1.20.1), Desktop Client (1.15.0), and App (1.20.0), along with a 'License & Labelling' link. At the bottom of the sidebar is a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area is divided into three sections: 'Patient', 'Notes', and 'Device setup'. The 'Patient' section contains a 'GDT patient file' upload area (highlighted with a red box) and form fields for 'Patient ID', 'Date of birth' (mm/dd/yyyy), 'First name', 'Last name', 'Gender' (Female, Male, Other), and 'Reason for recording' (Not selected). The 'Notes' section has a text area for 'Notes on patient or recording'. The 'Device setup' section includes dropdowns for 'Recording length' (1 day), 'Pacemaker' (None), and 'Sample rate' (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. On the right edge, a partial view of a medical device screen shows 'um C3+' and a battery level of 100%.

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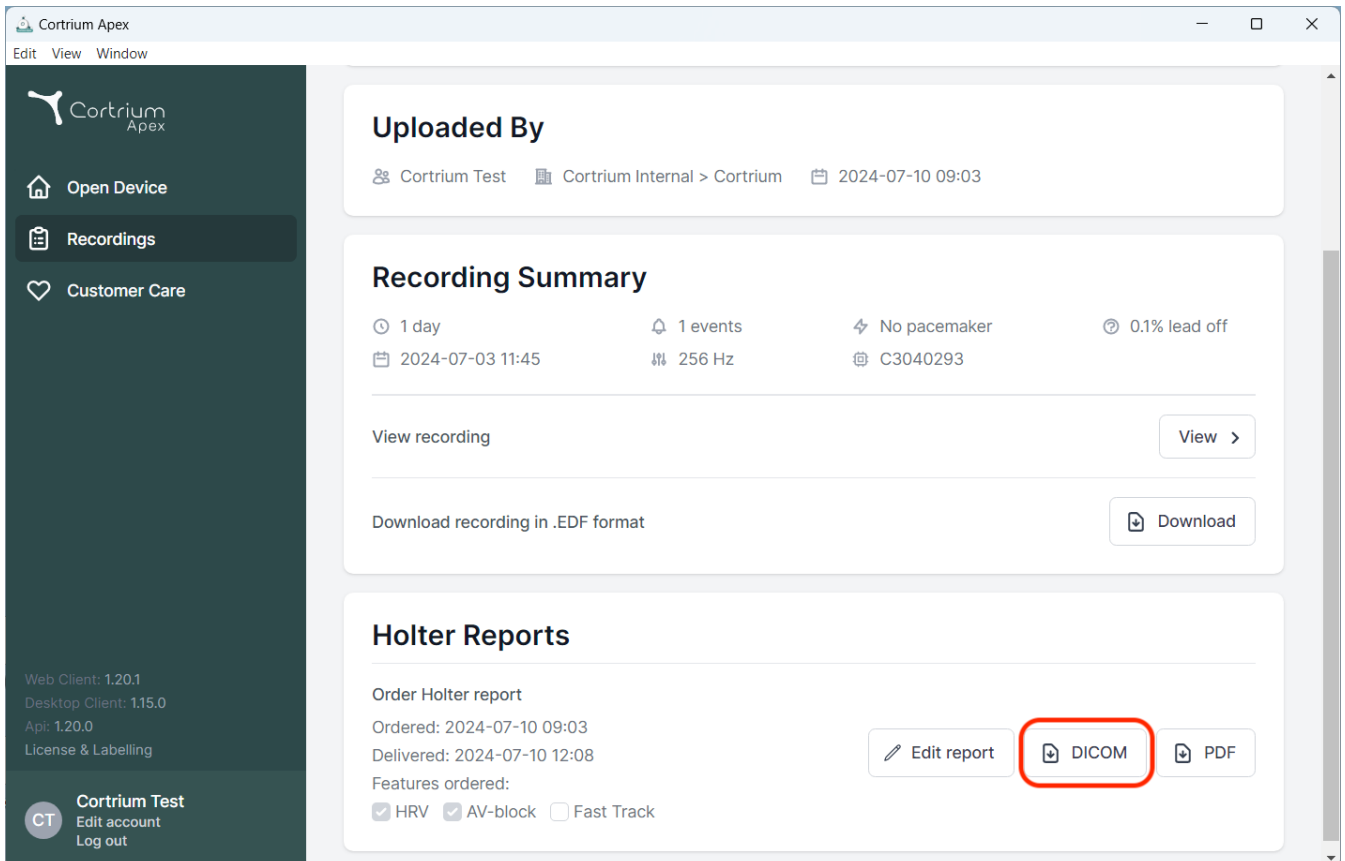The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By', 'Recording Summary', and 'Holter Reports'.

Uploaded By
Cortrium Test | Cortrium Internal > Cortrium | 2024-07-10 09:03


Recording Summary
🕒 1 day | 📅 2024-07-03 11:45 | 🔔 1 events | 📶 256 Hz | ⚡ No pacemaker | 📶 C3040293 | ⚙️ 0.1% lead off

Holter Reports
Order Holter report
Ordered: 2024-07-10 09:03
Delivered: 2024-07-10 12:08
Features ordered:
☒ HRV ☒ AV-block ☐ Fast Track

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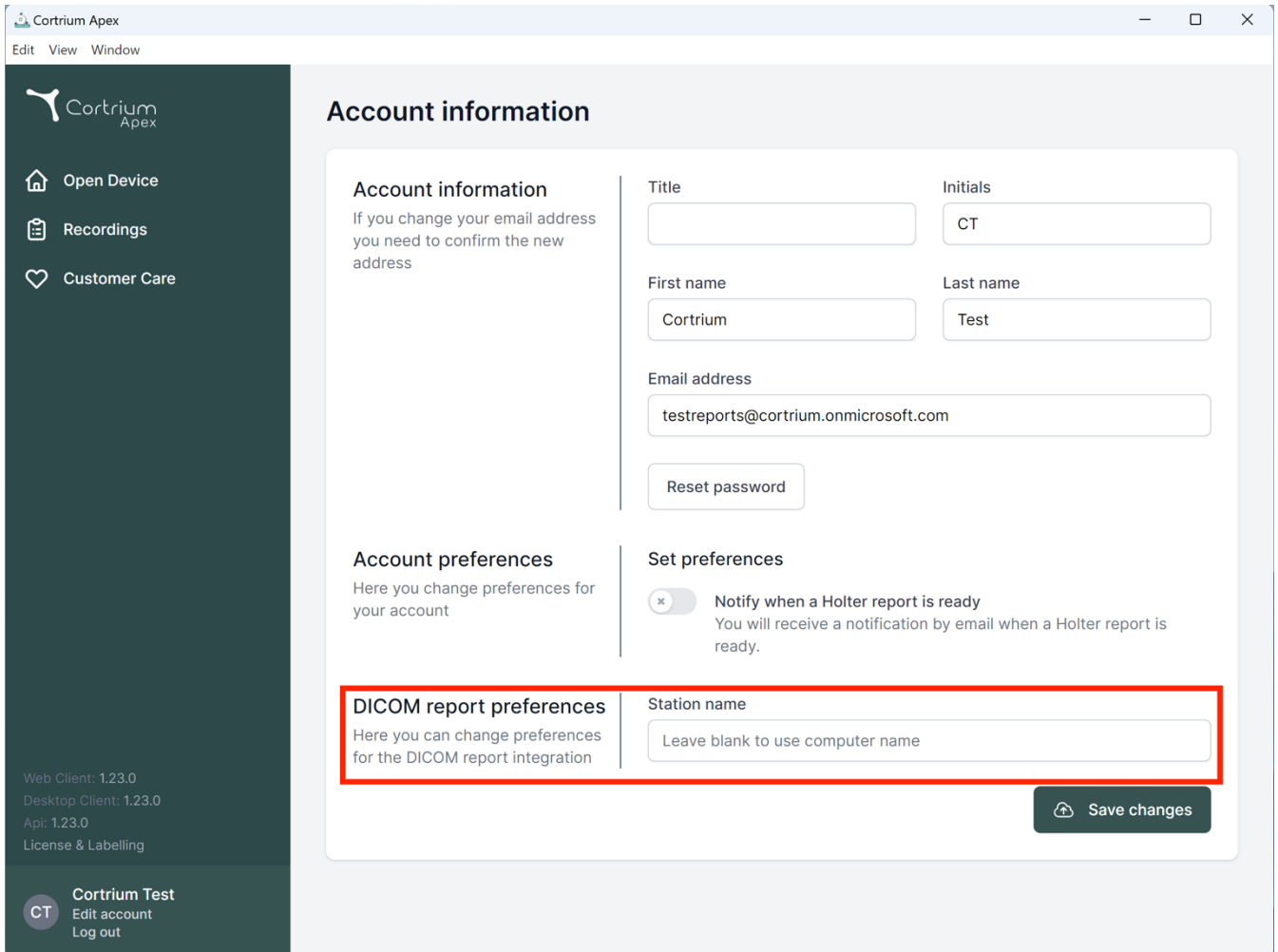Account information

If you change your email address you need to confirm the new address

Account information

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

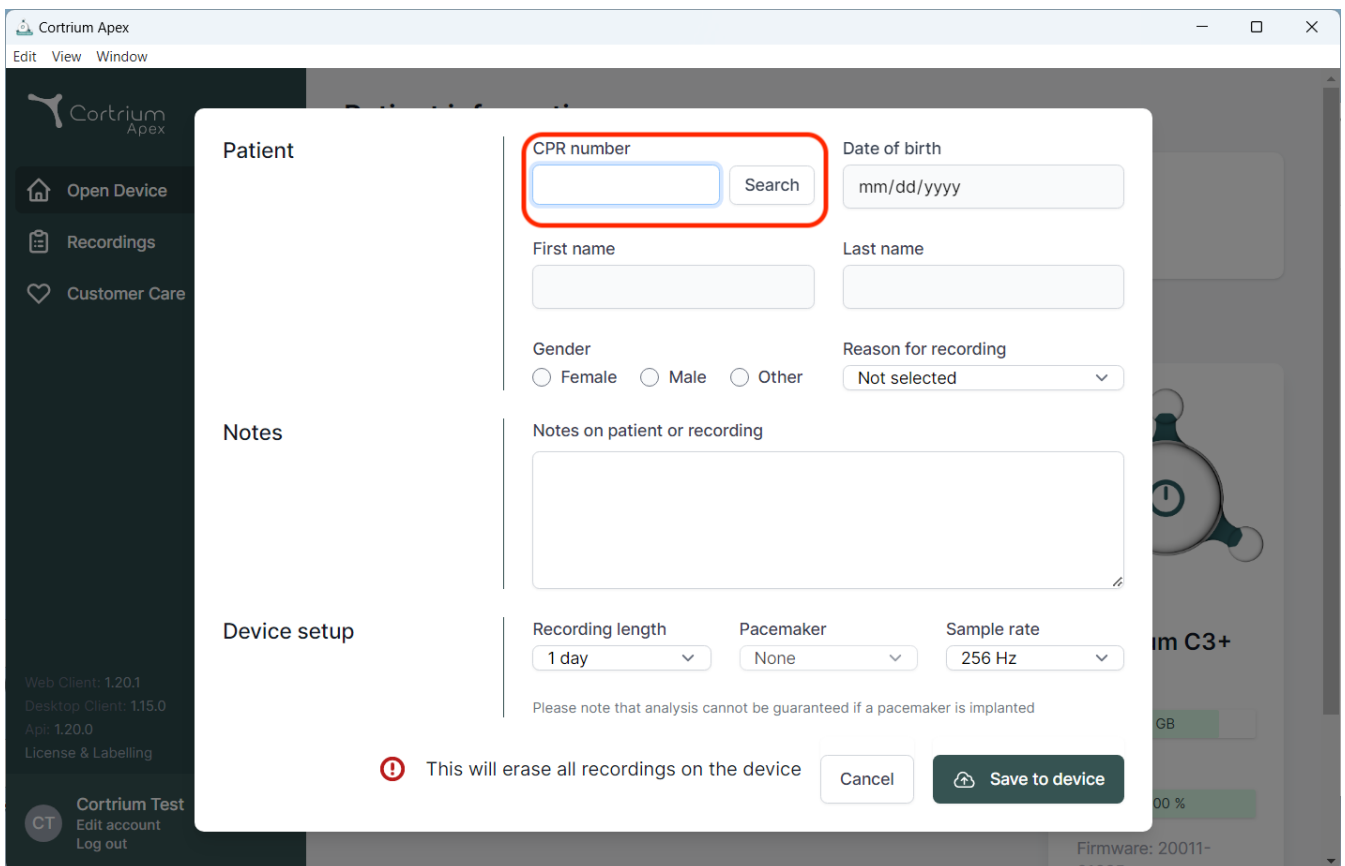
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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
The screenshot shows the Cortrium Apex web application interface. A modal form is displayed with the following sections:

- Patient**
 - CPR number**: A text input field with a red border and a "Search" button next to it.
 - Date of birth**: A text input field with a placeholder "mm/dd/yyyy".
 - First name**: A text input field.
 - Last name**: A text input field.
 - Gender**: Radio buttons for "Female", "Male", and "Other".
 - Reason for recording**: A dropdown menu with "Not selected" as the current option.
- Notes**
 - Notes on patient or recording**: A large text area for notes.
- Device setup**
 - Recording length**: A dropdown menu with "1 day" selected.
 - Pacemaker**: A dropdown menu with "None" selected.
 - Sample rate**: A dropdown menu with "256 Hz" selected.

At the bottom of the form, there is a warning message: "This will erase all recordings on the device" with a red exclamation mark icon. Below this message are two buttons: "Cancel" and "Save to device".

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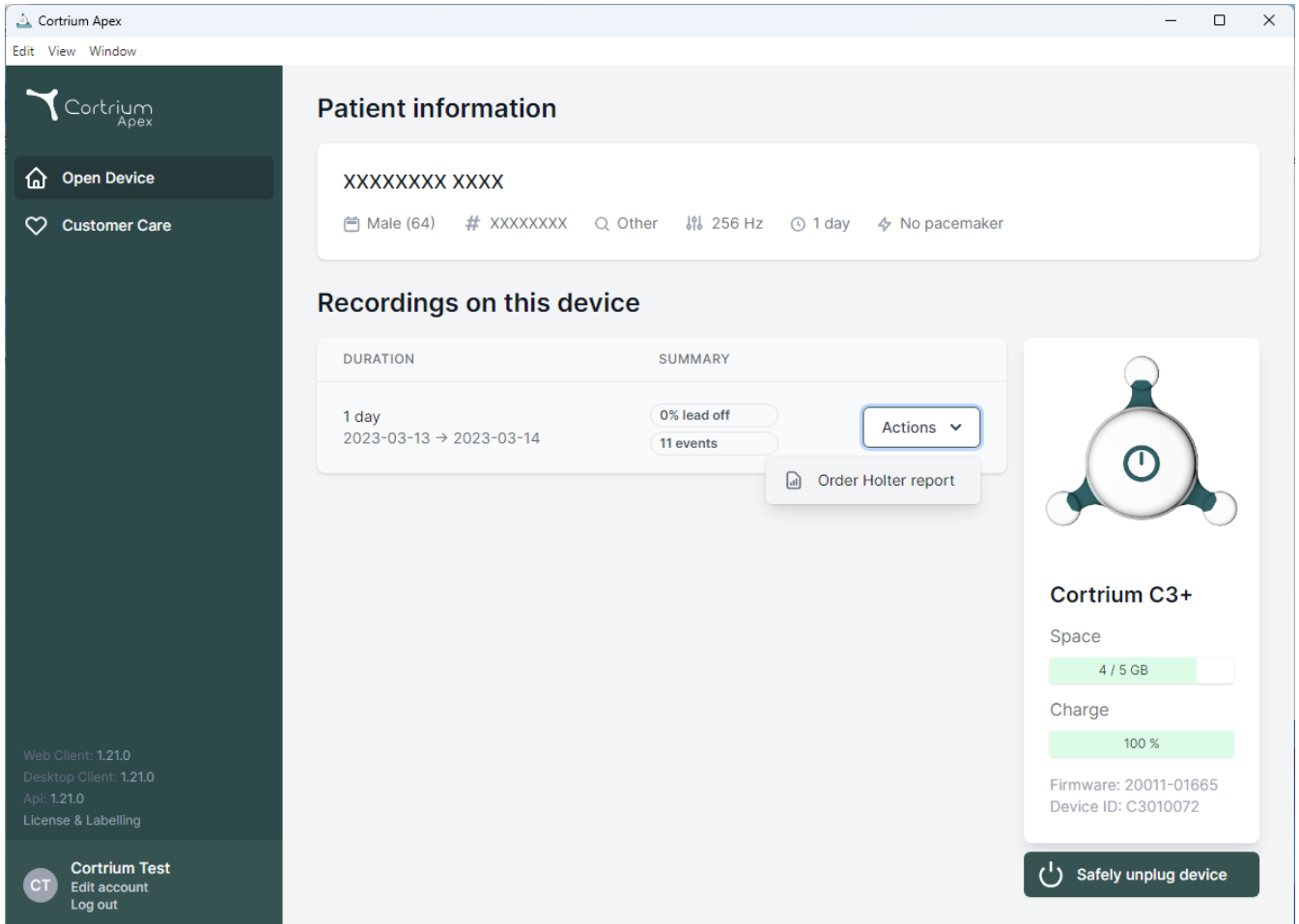
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device' (with a house icon), 'Customer Care' (with a heart icon), and version information: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar is a 'Cortrium Test' section with a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section contains a table with two columns: 'DURATION' and 'SUMMARY'. The table has one entry for '1 day' (2023-03-13 → 2023-03-14) with a summary of '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the summary, with an option to 'Order Holter report'. On the right side, there is a device status card for 'Cortrium C3+'. It shows a battery icon, 'Space' (4 / 5 GB), 'Charge' (100 %), 'Firmware: 20011-01665', and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

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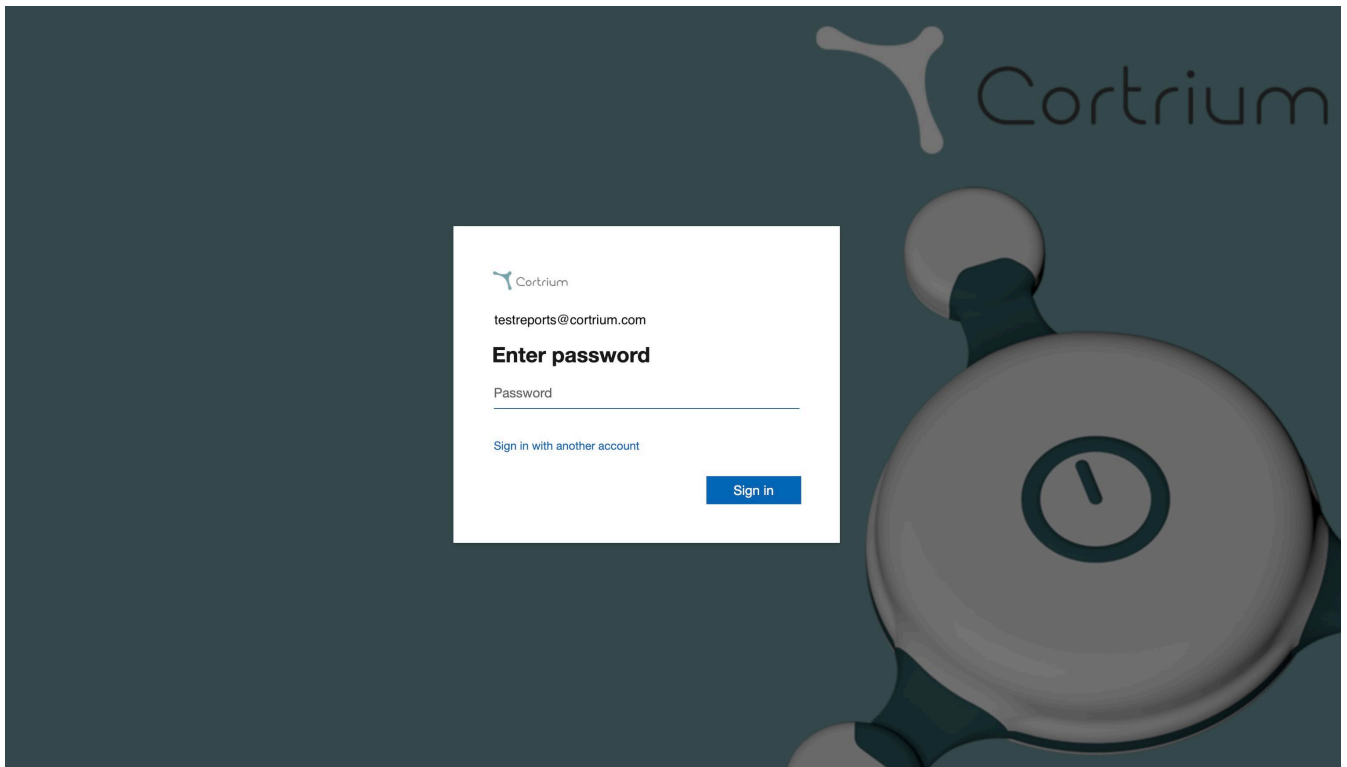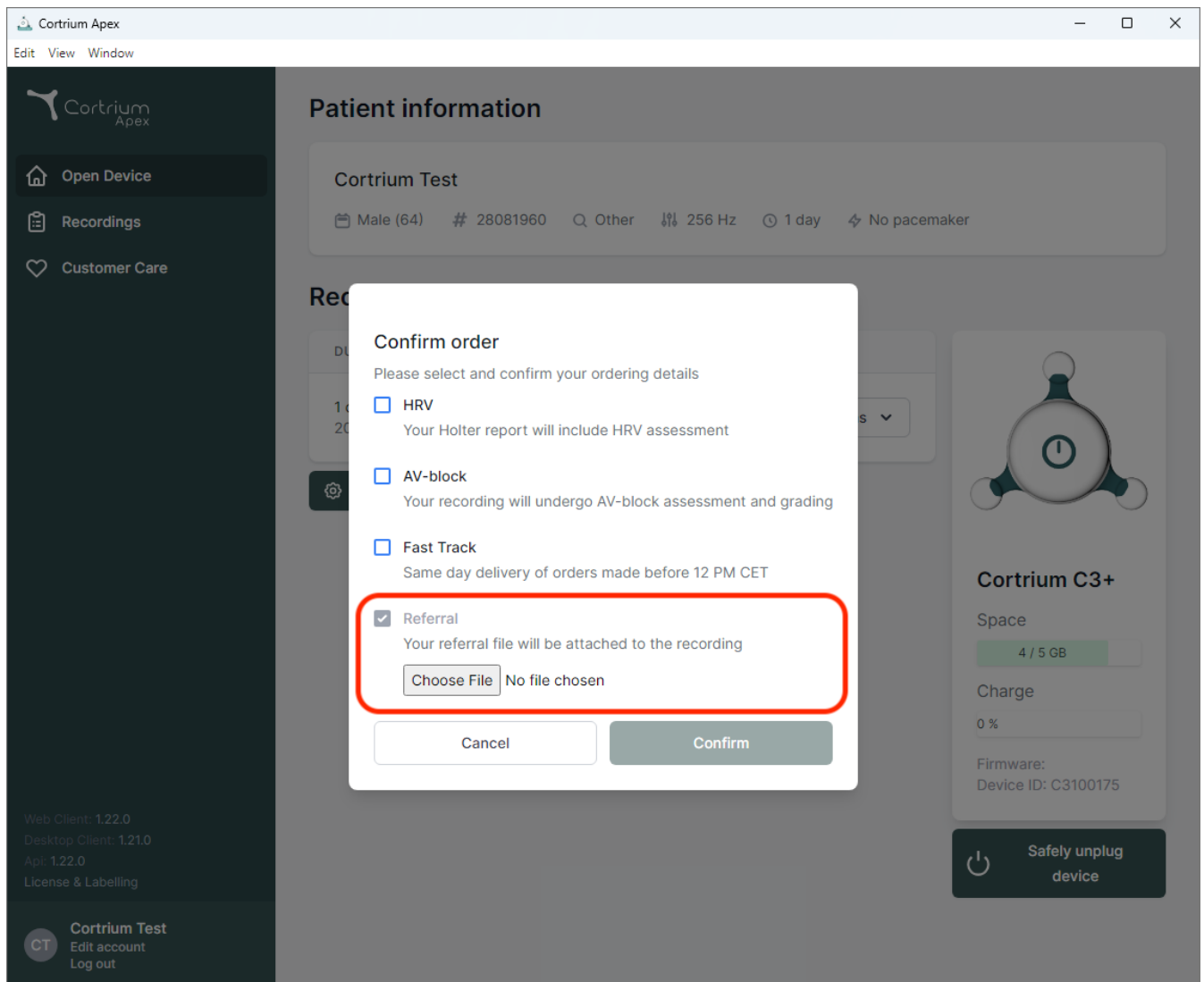
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
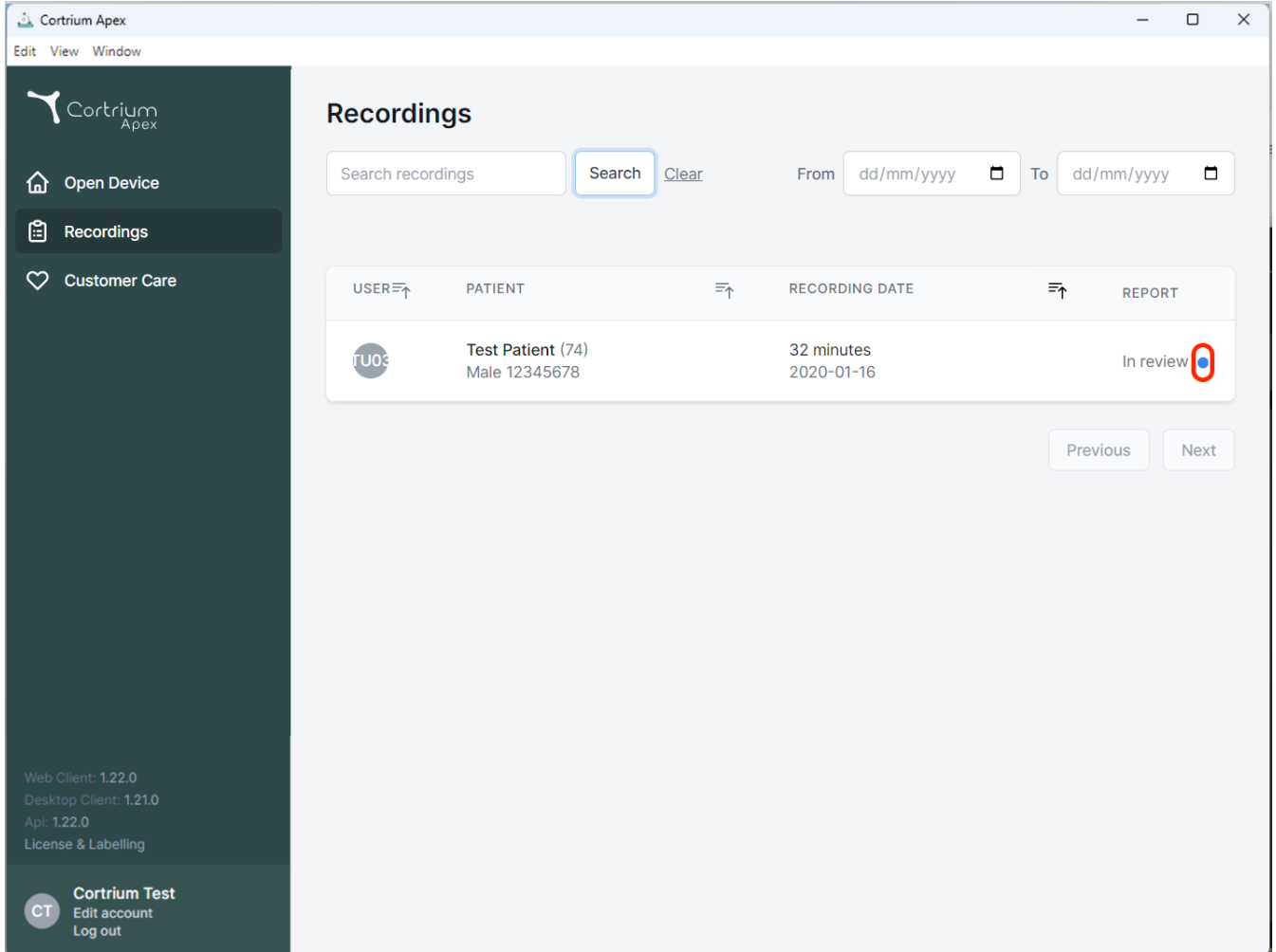
The screenshot shows the 'Recordings' section of the Cortrium Apex web application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main area displays a table of recordings with columns for USER, PATIENT, RECORDING DATE, and REPORT. A single recording is listed for 'Test Patient (74)' with a duration of '32 minutes' and a date of '2020-01-16'. The 'REPORT' column shows 'In review' with a blue dot icon circled in red. Search and date filters are available at the top, and 'Previous' and 'Next' buttons are at the bottom right.

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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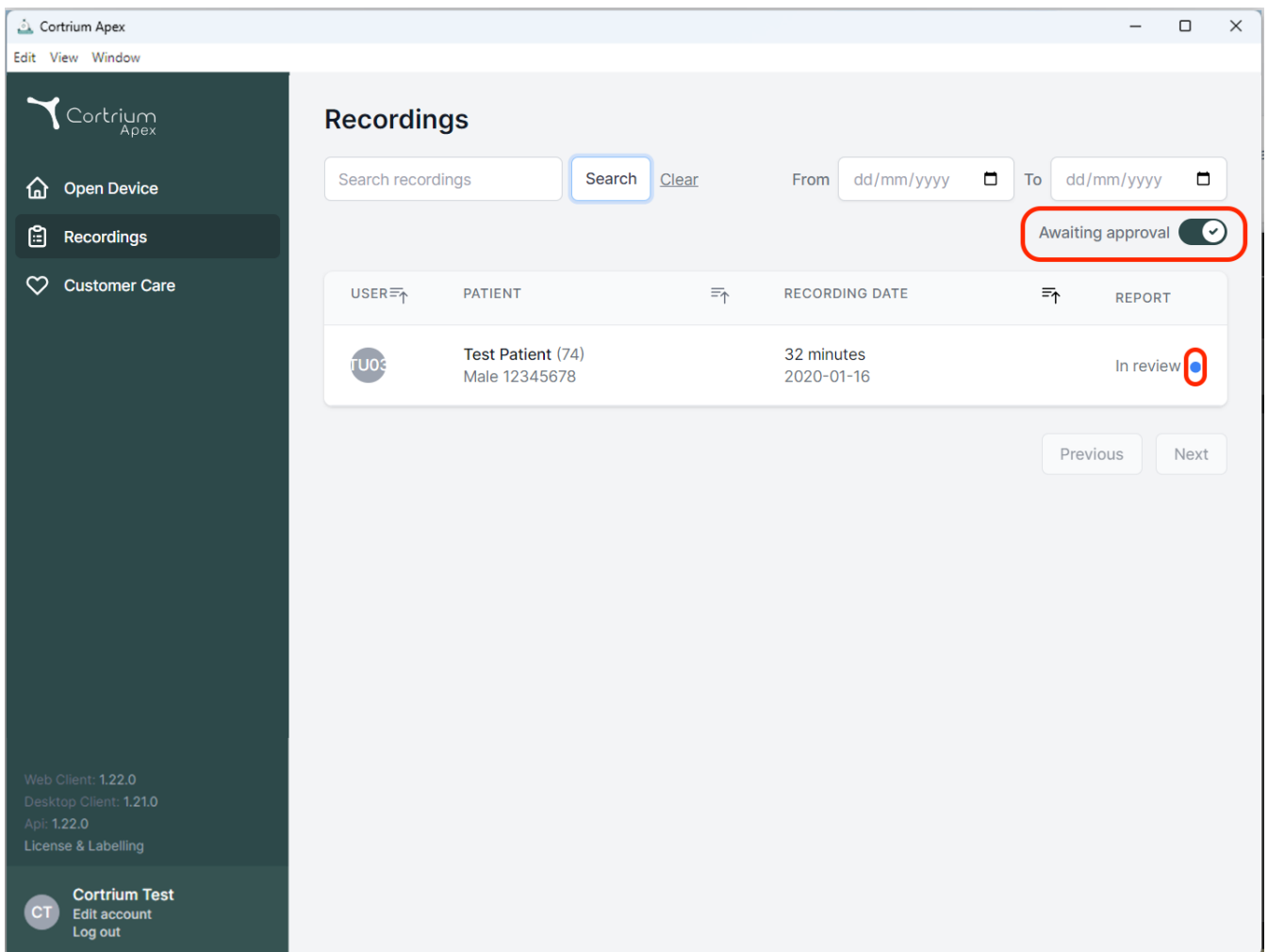Search recordings From To


Awaiting approval ☒

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review ●

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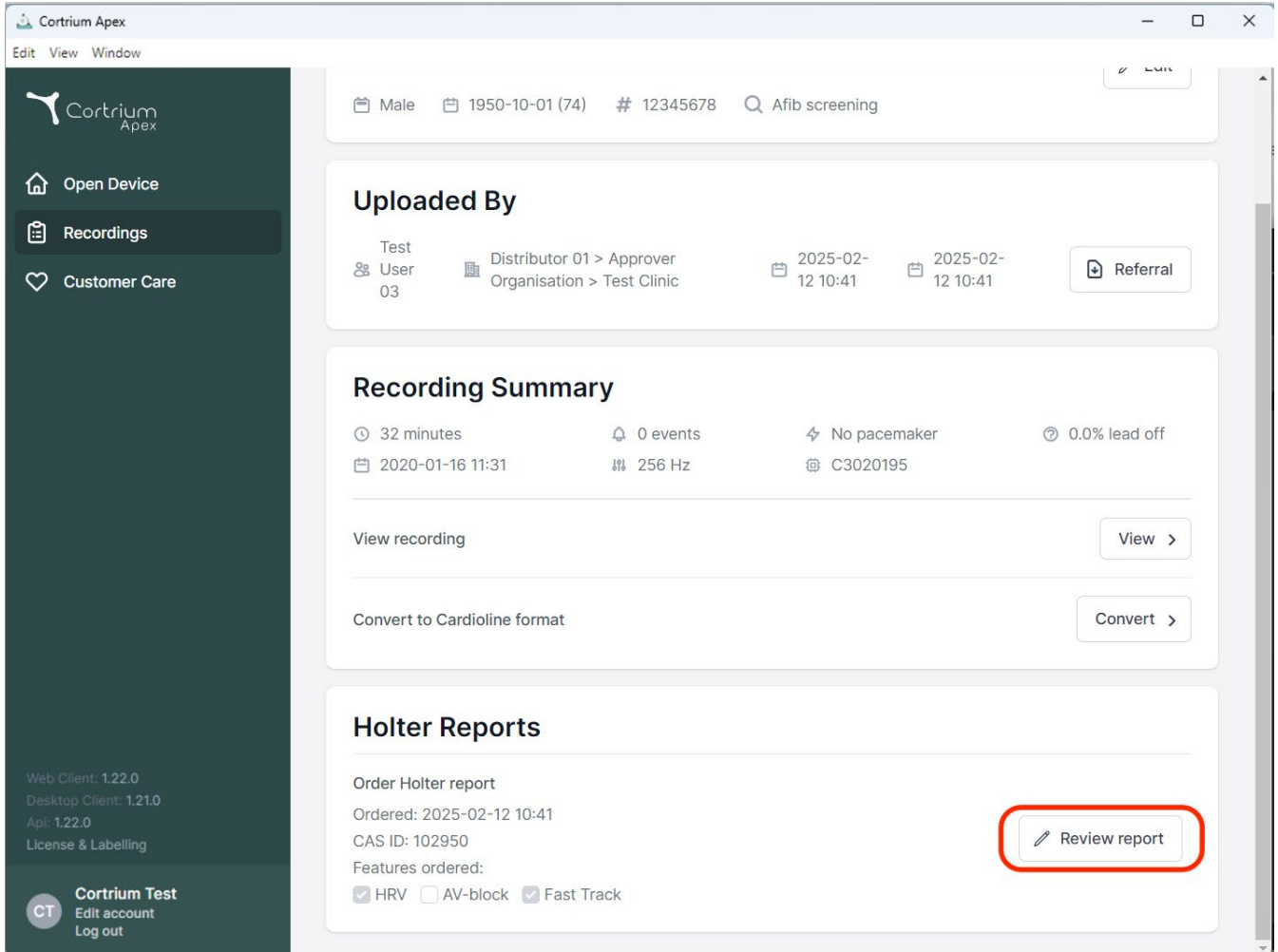
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with timestamps '2025-02-12 10:41' and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

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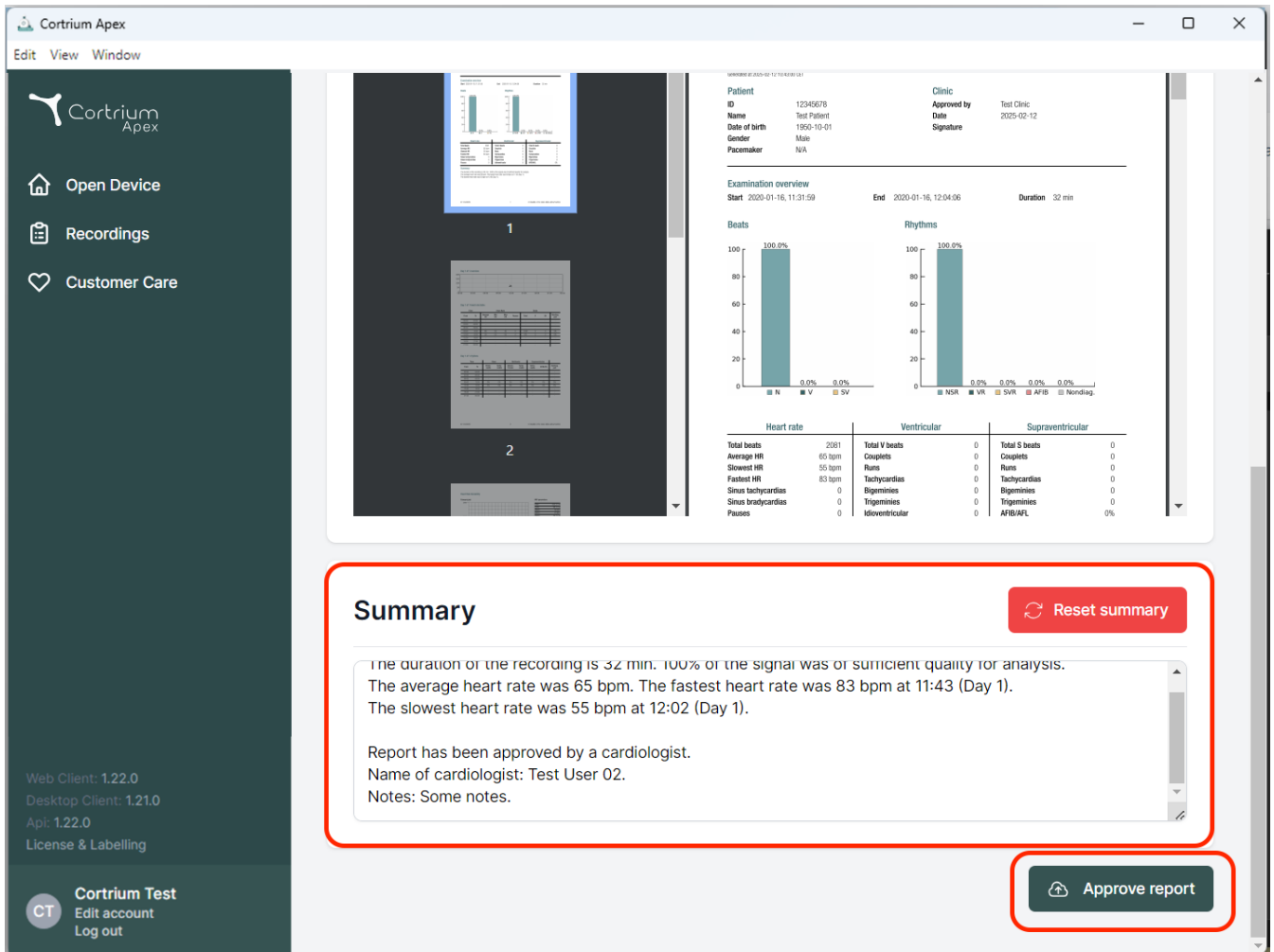Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

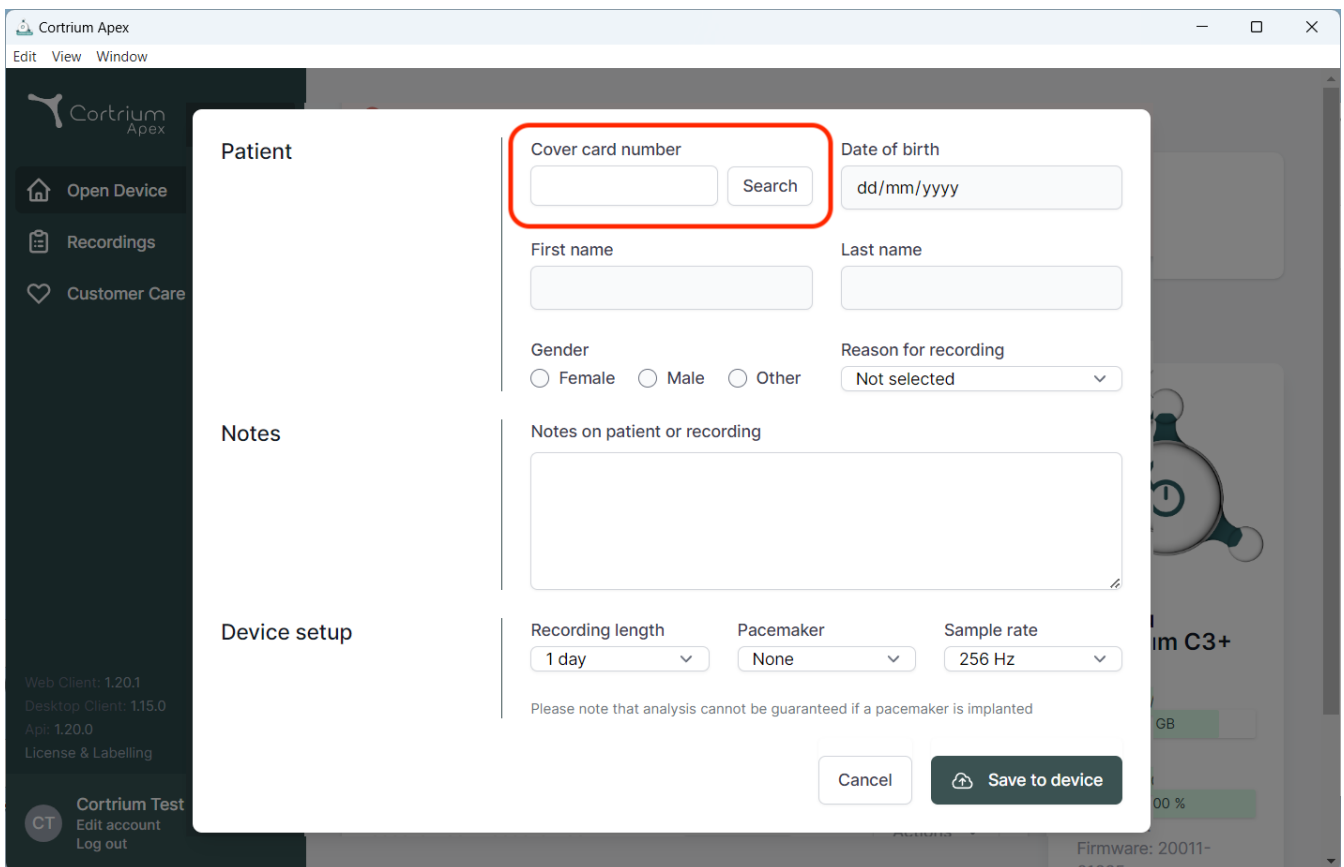
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient search and device setup. The form is divided into three main sections: Patient, Notes, and Device setup.

Patient Section:

- Cover card number:** A text input field with a "Search" button next to it. This section is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu with "1 day" as the selected option.
- Pacemaker:** A dropdown menu with "None" as the selected option.
- Sample rate:** A dropdown menu with "256 Hz" as the selected option.

Below the device setup section, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the form, there are two buttons: "Cancel" and "Save to device".

The background of the application shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar contains version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right corner of the sidebar shows a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark
Email: info@cortrium.com • Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47