Cortrium Apex 1.31

Kasutusjuhend

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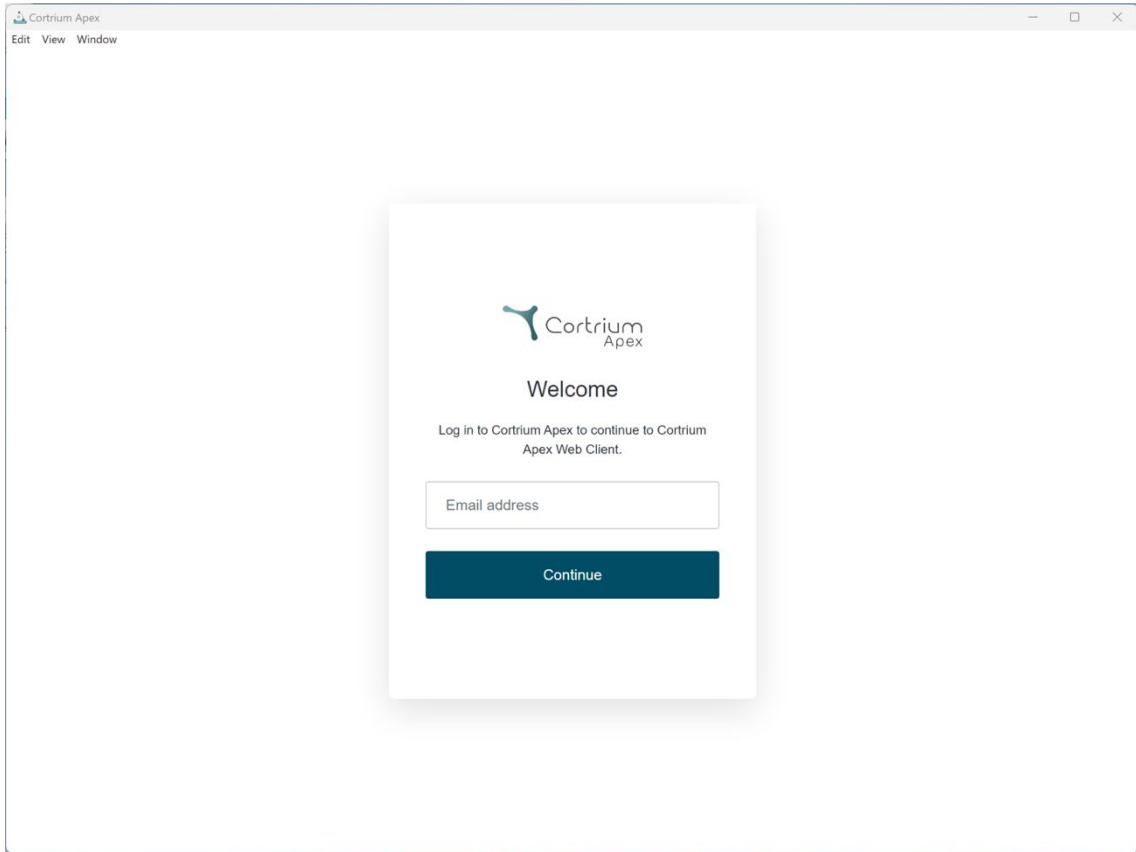
 Cortrium
Apex

Welcome

Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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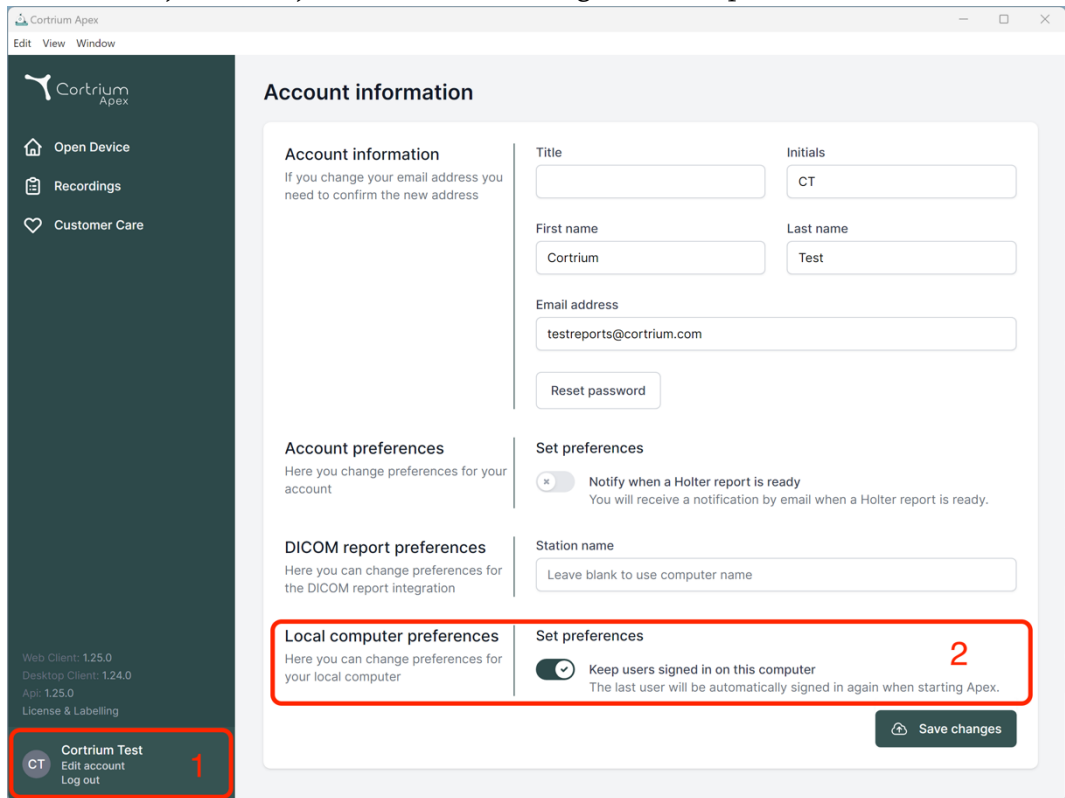
on

Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

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DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

Local computer preferences

Here you can change preferences for your local computer

☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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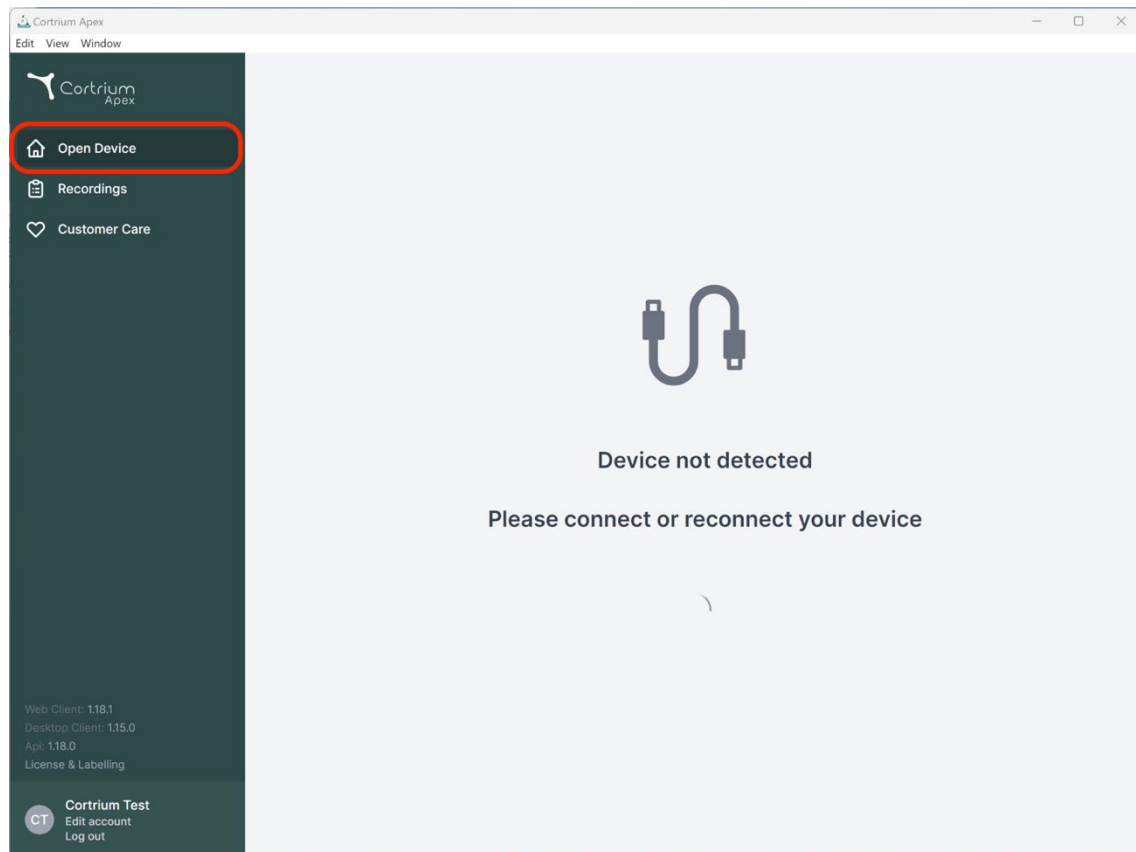
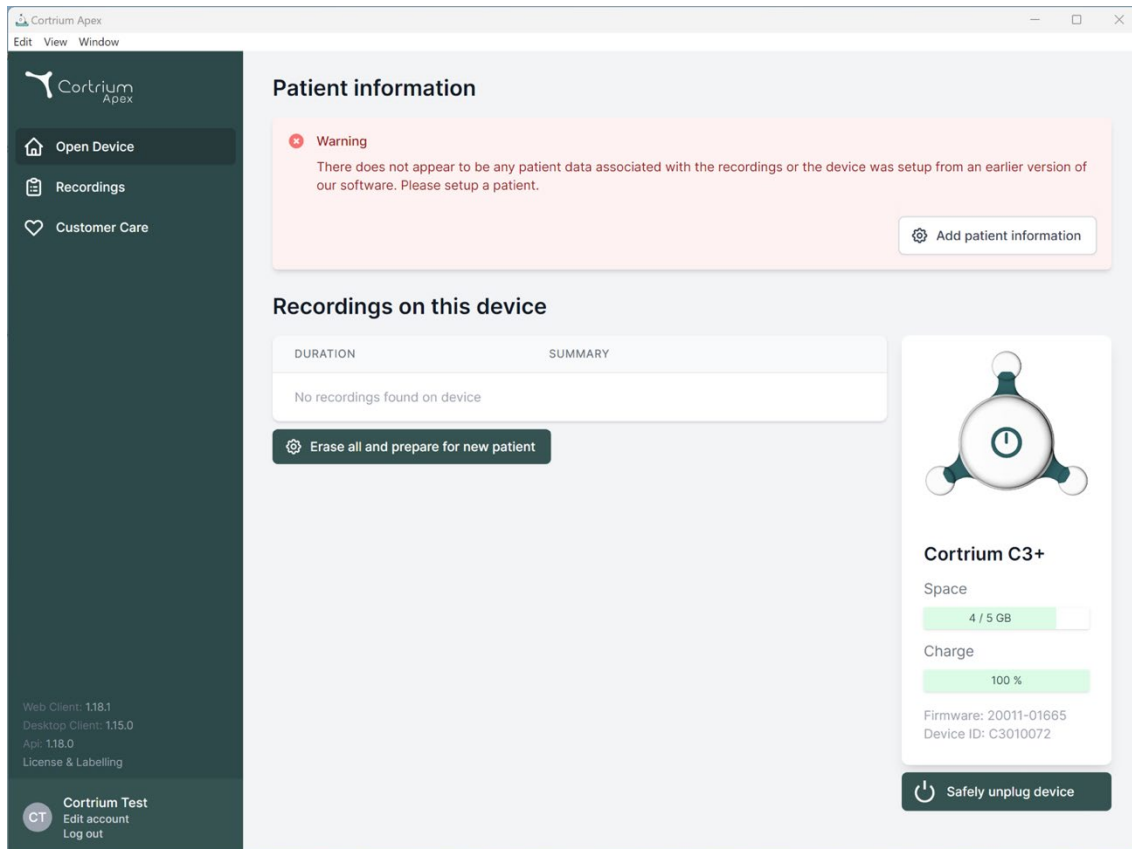
Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

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Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Save to device

Warning This will erase all recordings on the device

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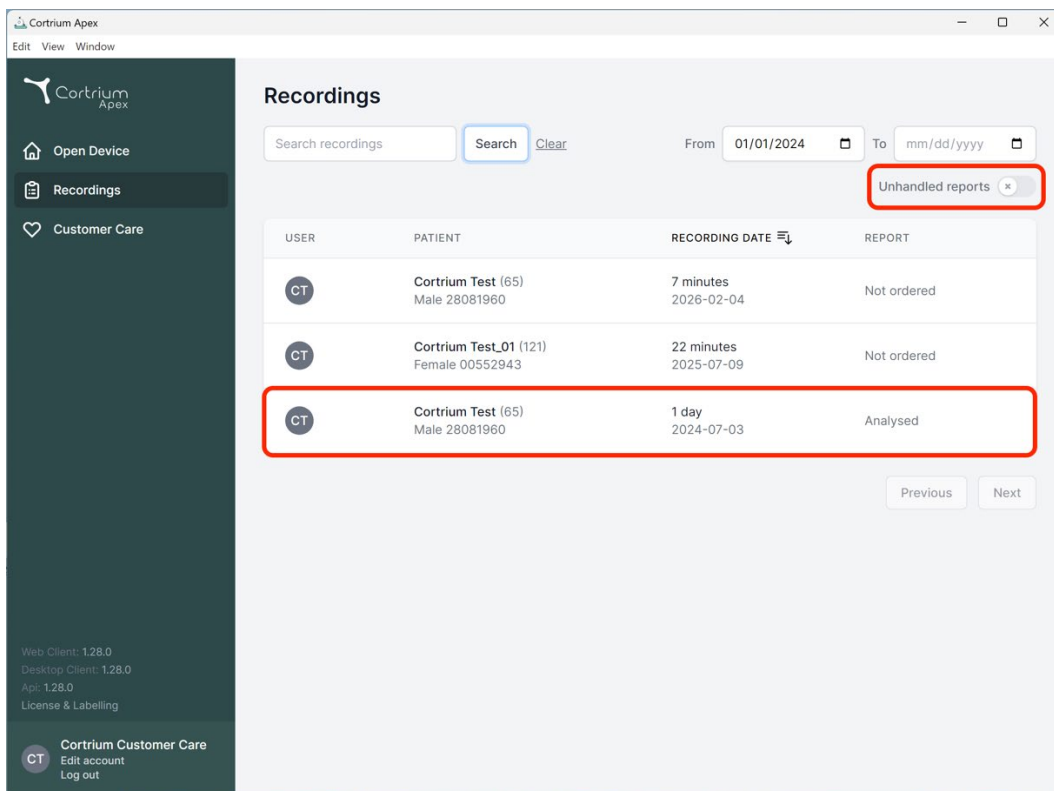
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main area is titled 'Recordings' and contains a search bar, a 'Search' button, a 'Clear' link, and date range filters. A dropdown menu labeled 'Unhandled reports' is highlighted with a red box. Below this is a table with the following data:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (65) Male 28081960	7 minutes 2026-02-04	Not ordered
CT	Cortrium Test_01 (121) Female 00552943	22 minutes 2025-07-09	Not ordered
CT	Cortrium Test (65) Male 28081960	1 day 2024-07-03	Analysed

The third row of the table is highlighted with a red box. At the bottom right of the table are 'Previous' and 'Next' buttons. The sidebar at the bottom shows version information (Web Client: 1.28.0, Desktop Client: 1.28.0, Api: 1.28.0) and a 'Cortrium Customer Care' section with links for 'Edit account' and 'Log out'.

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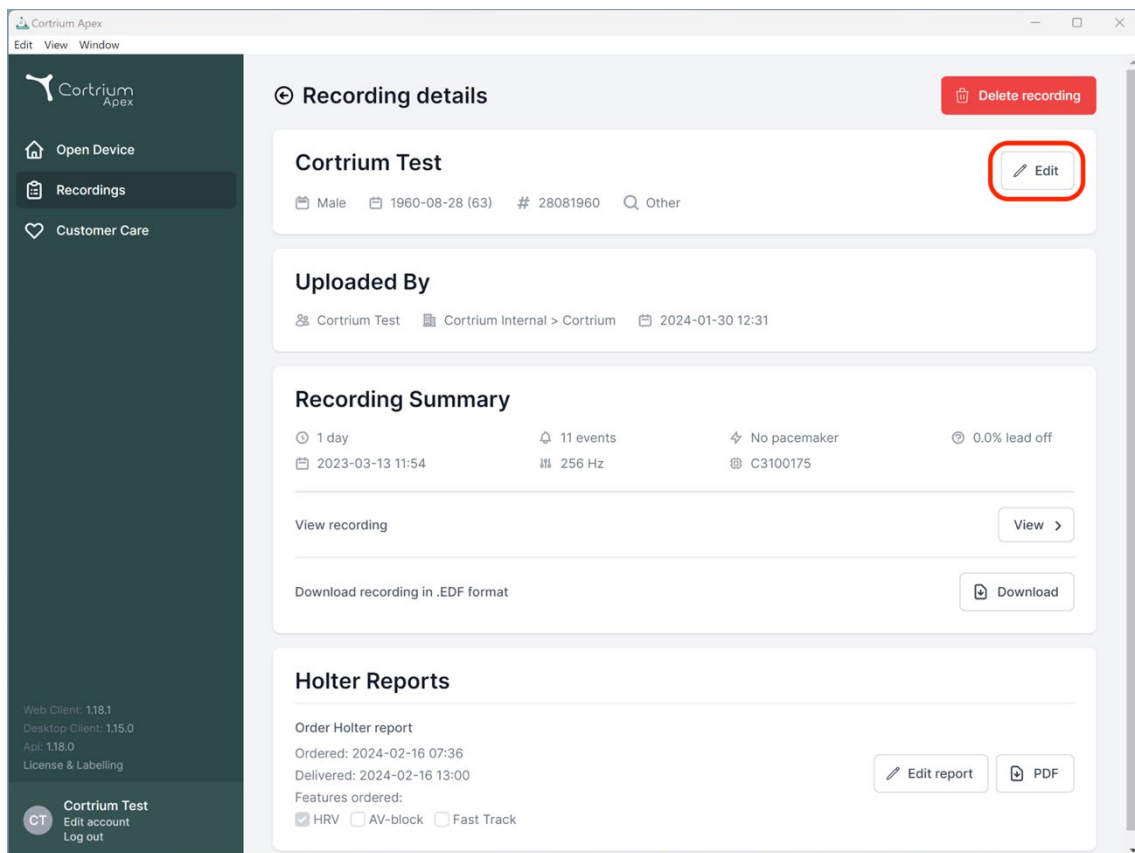
Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

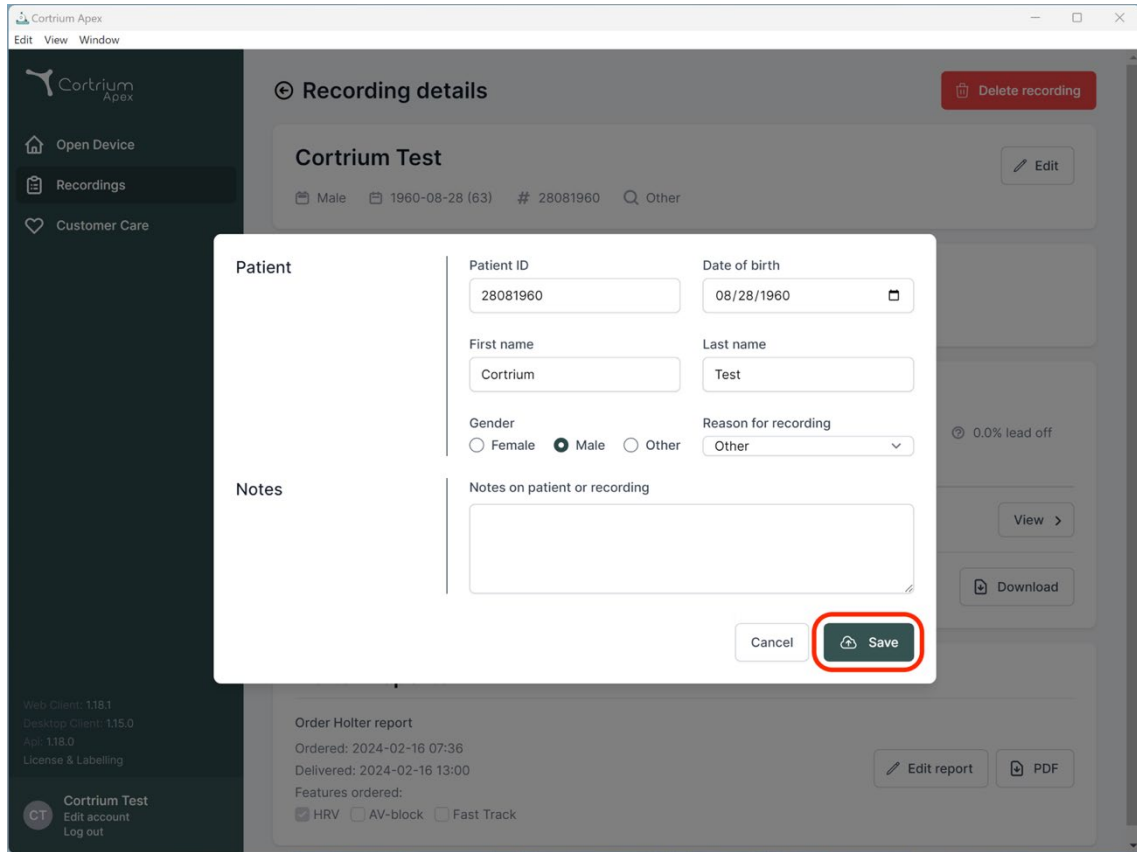
Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Cancel Save

Web Client: 118.1 Desktop Client: 115.0 App: 118.0 License & Labelling

Order Holter report

Ordered: 2024-02-16 07:36 Delivered: 2024-02-16 13:00

Features ordered: ☒ HRV ☐ AV-block ☐ Fast Track

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
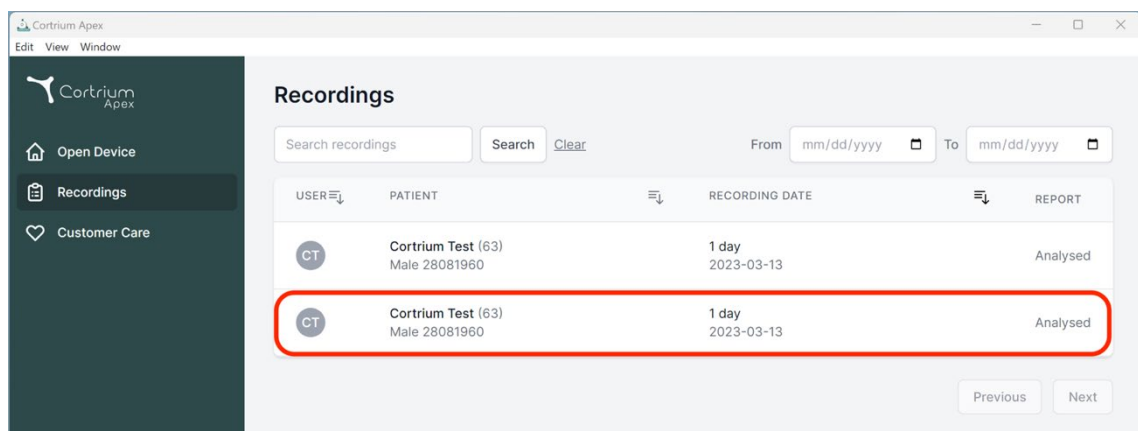
Recordings

Search recordings Search Clear From mm/dd/yyyy To mm/dd/yyyy

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

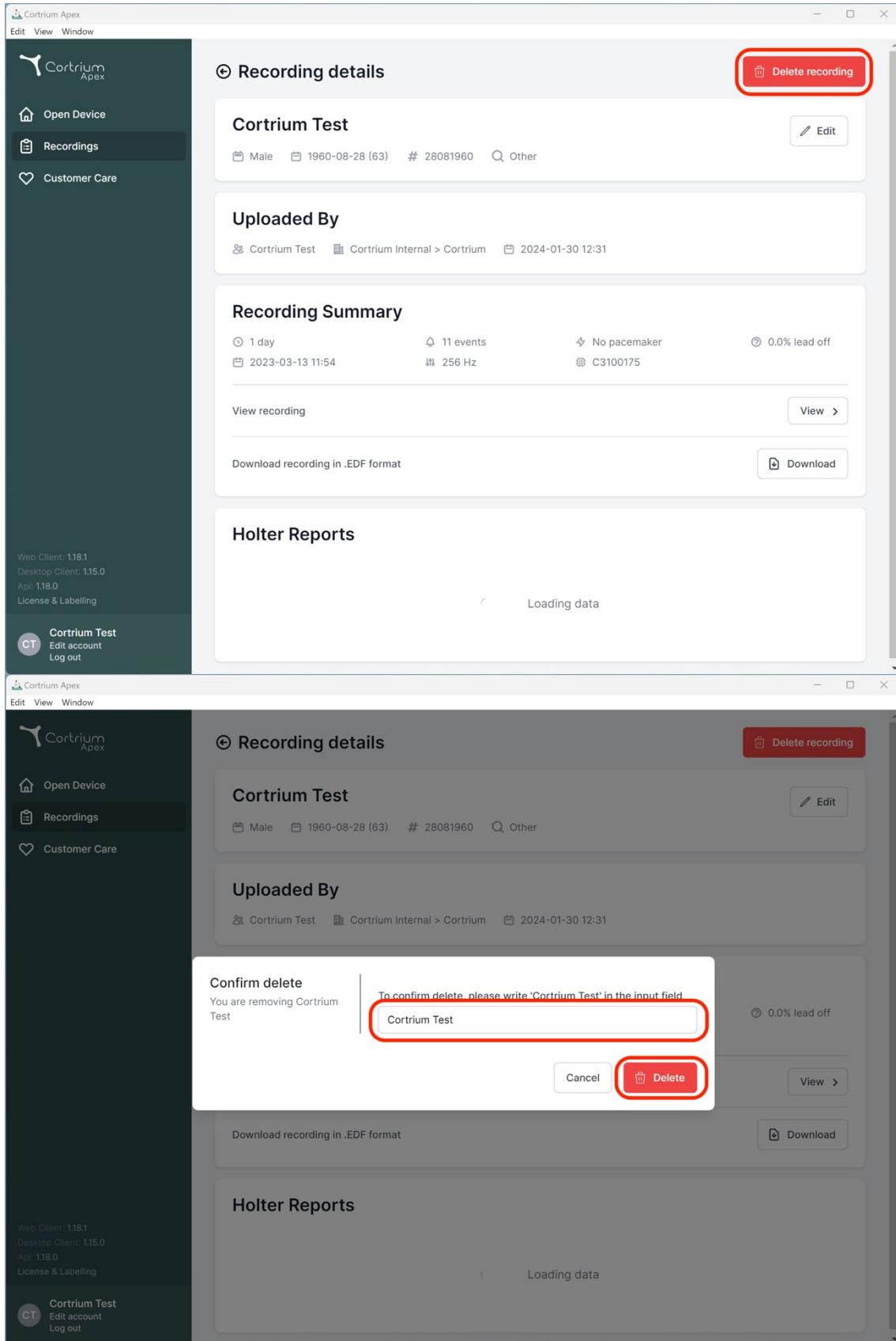Previous Next

Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Loading data

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

Cortrium Test

Cancel Delete

Download recording in .EDF format Download

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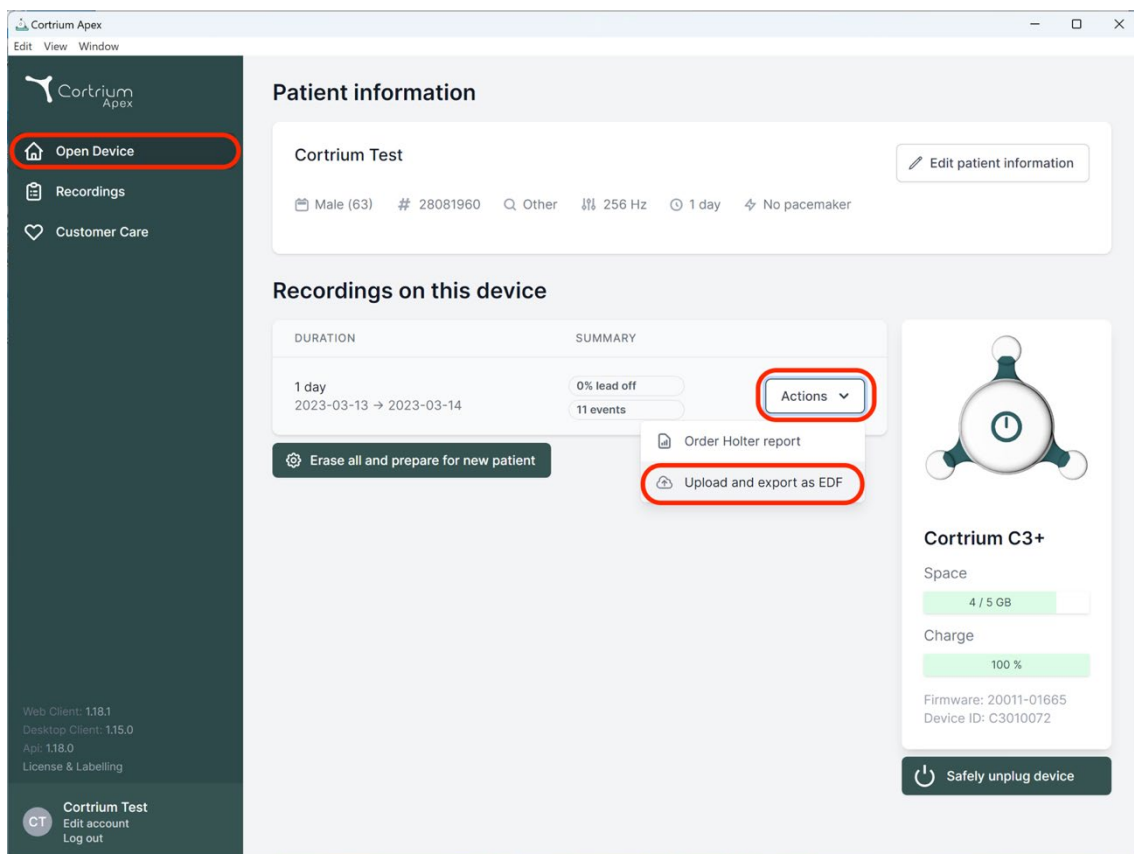
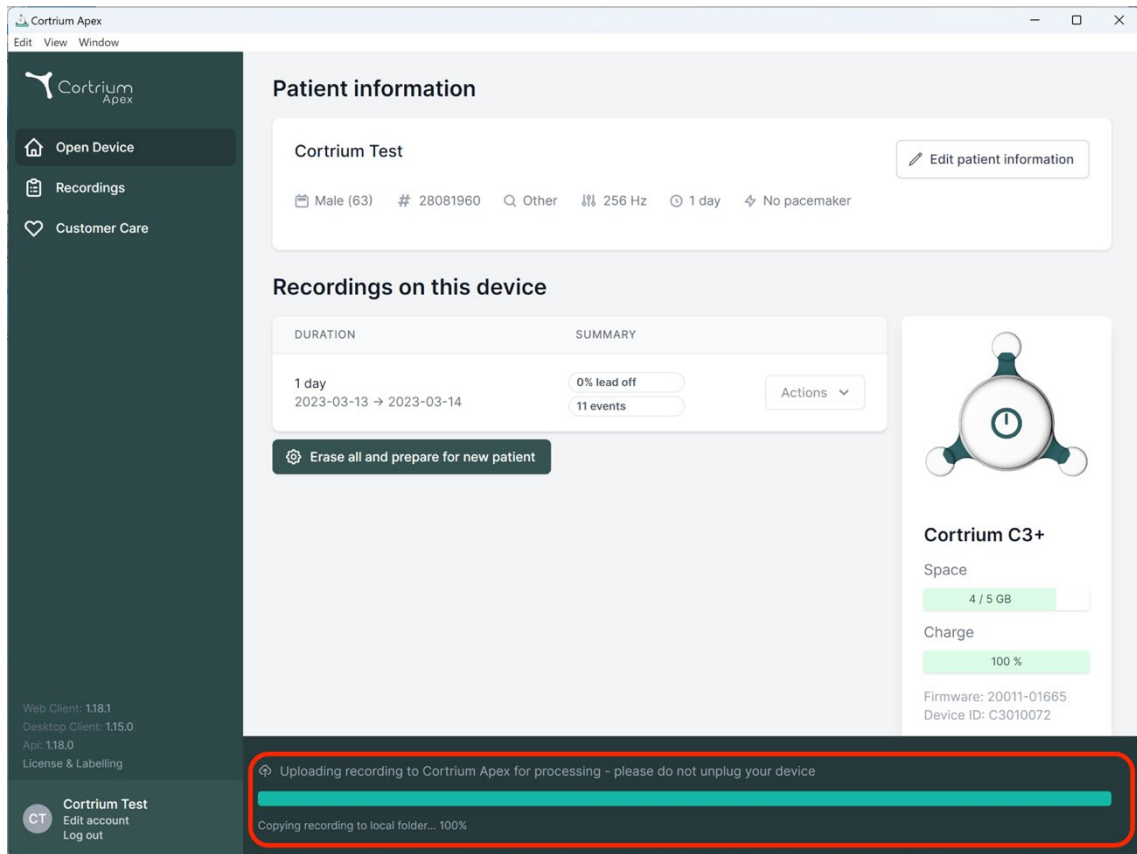
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

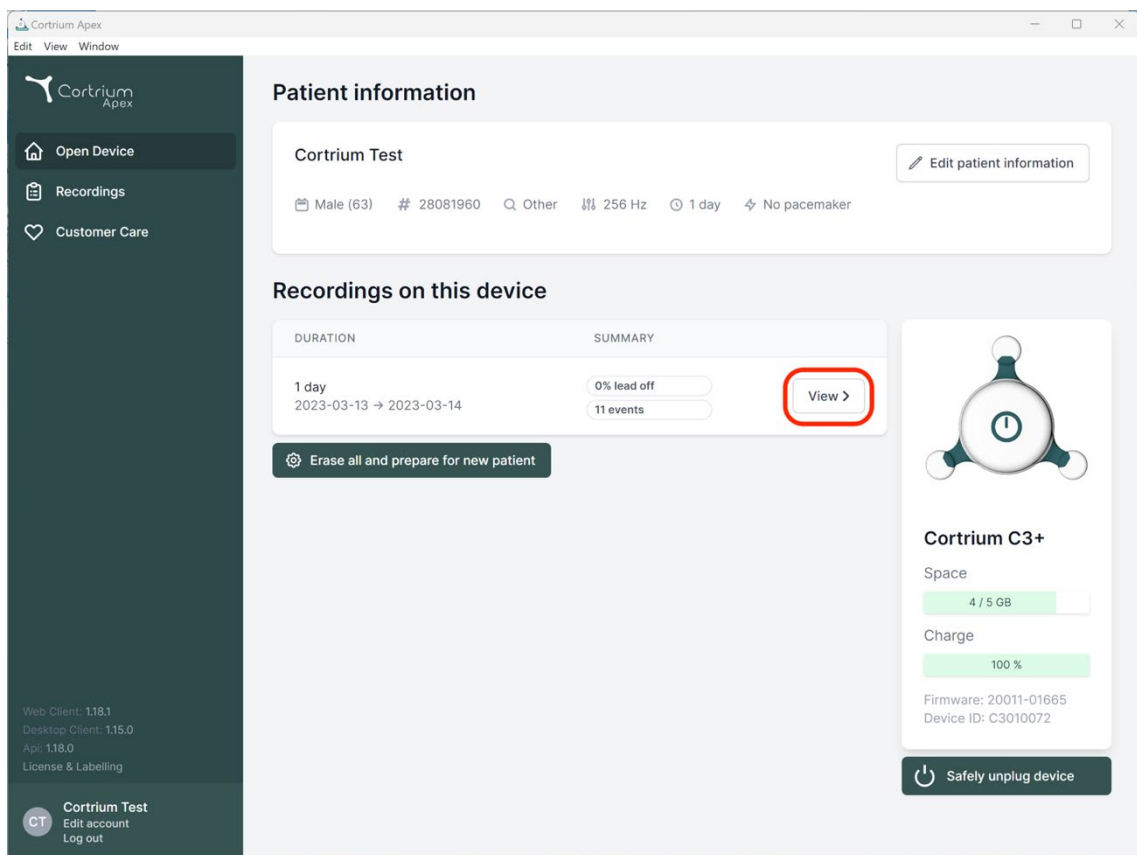
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
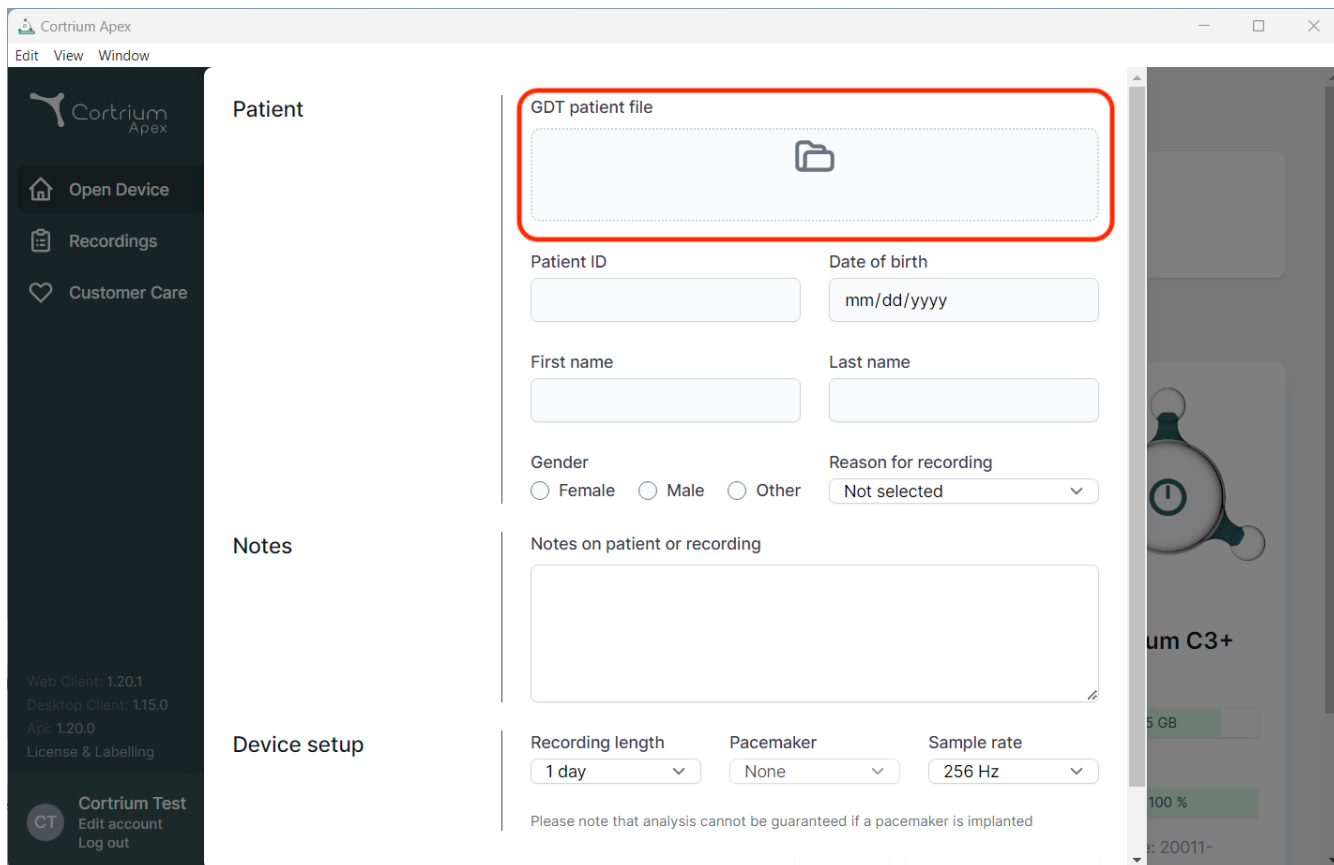
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", and "Api: 1.20.0". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options. The main content area is divided into three sections: "Patient", "Notes", and "Device setup". The "Patient" section contains a "GDT patient file" upload area (highlighted with a red box) and form fields for "Patient ID", "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), and "Reason for recording" (Not selected). The "Notes" section has a text area for "Notes on patient or recording". The "Device setup" section includes dropdowns for "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A disclaimer at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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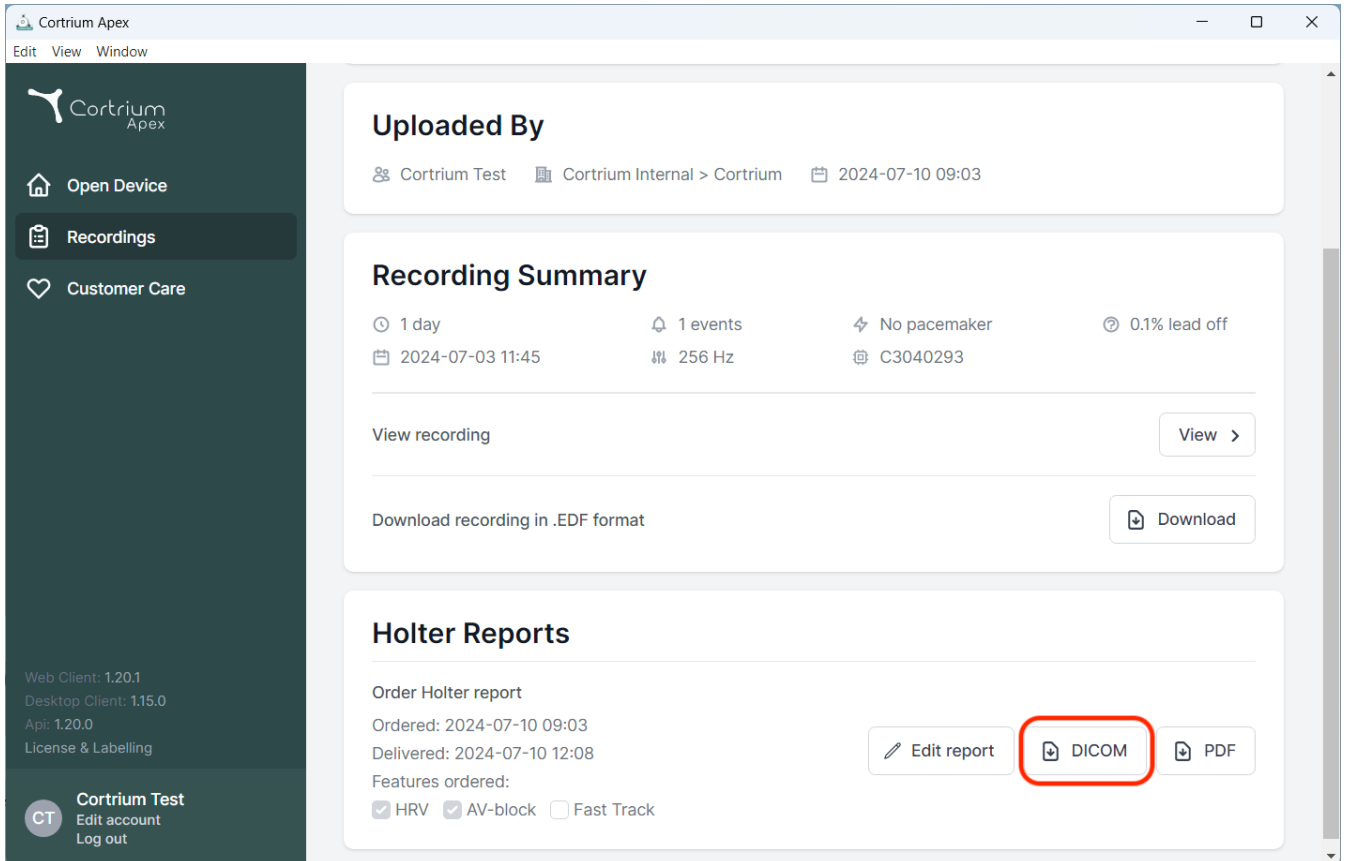
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (labeled 'Download recording in .EDF format').
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are buttons for 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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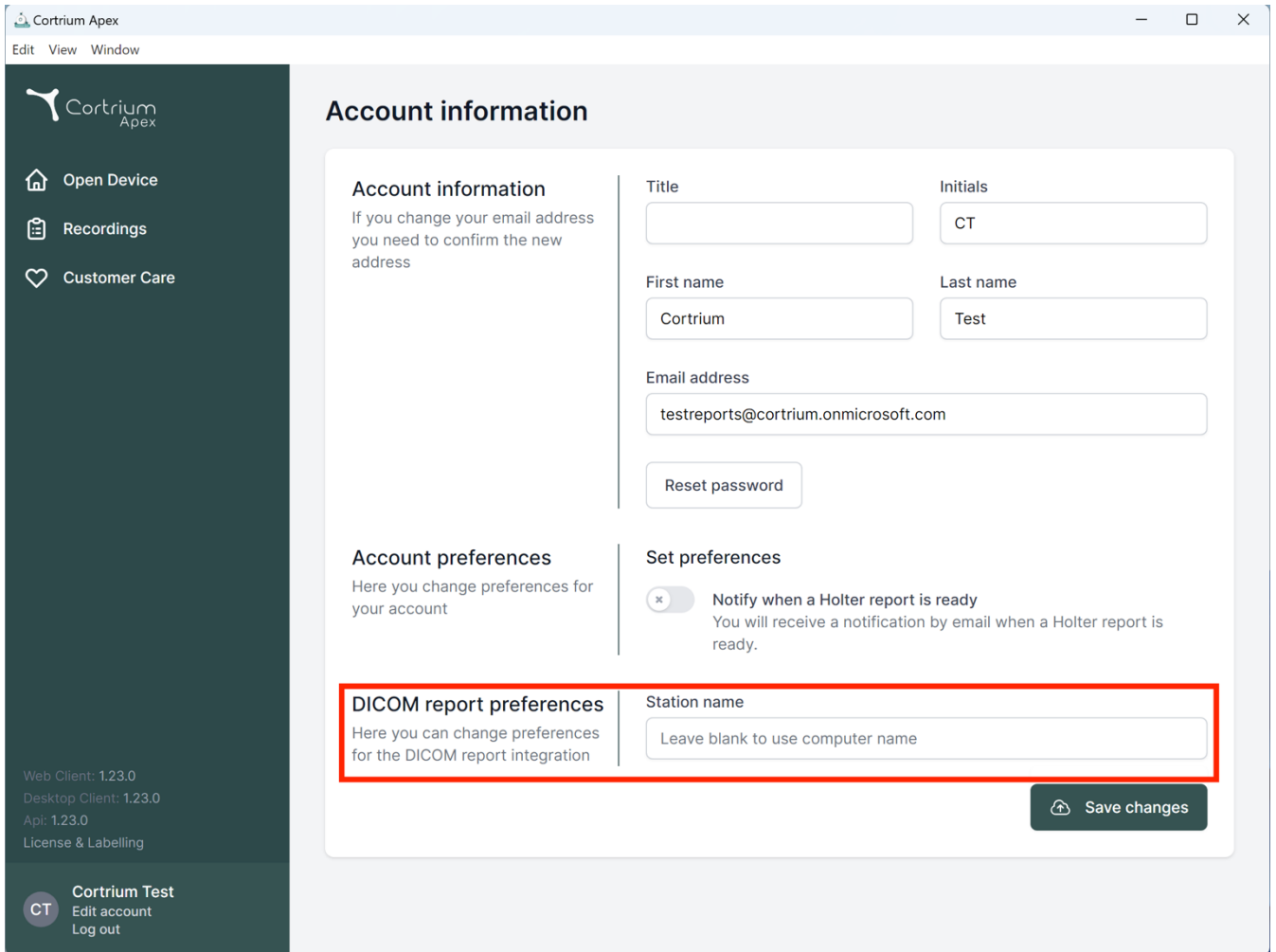Cortrium Apex
Edit View Window

Account information

Account information
If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name:

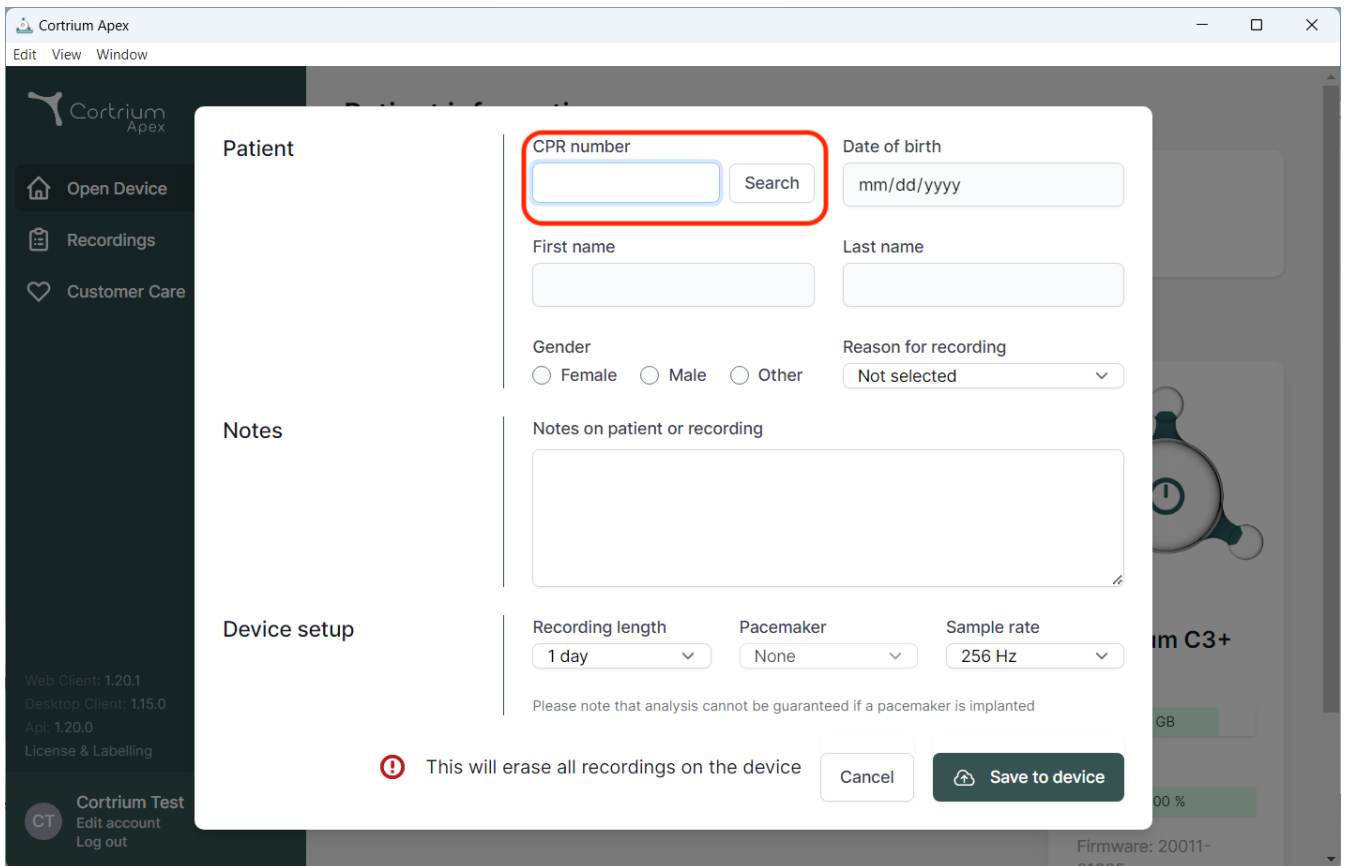
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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
The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying a search form. The form includes fields for "CPR number" (highlighted with a red box), "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), and "Reason for recording" (Not selected). Below these fields is a "Notes on patient or recording" text area. At the bottom of the form, there are "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz) dropdown menus. A warning message states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom of the modal, there is a red warning icon and the text "This will erase all recordings on the device", along with "Cancel" and "Save to device" buttons. The background shows the Cortrium Apex main interface with a sidebar containing "Open Device", "Recordings", and "Customer Care" options.

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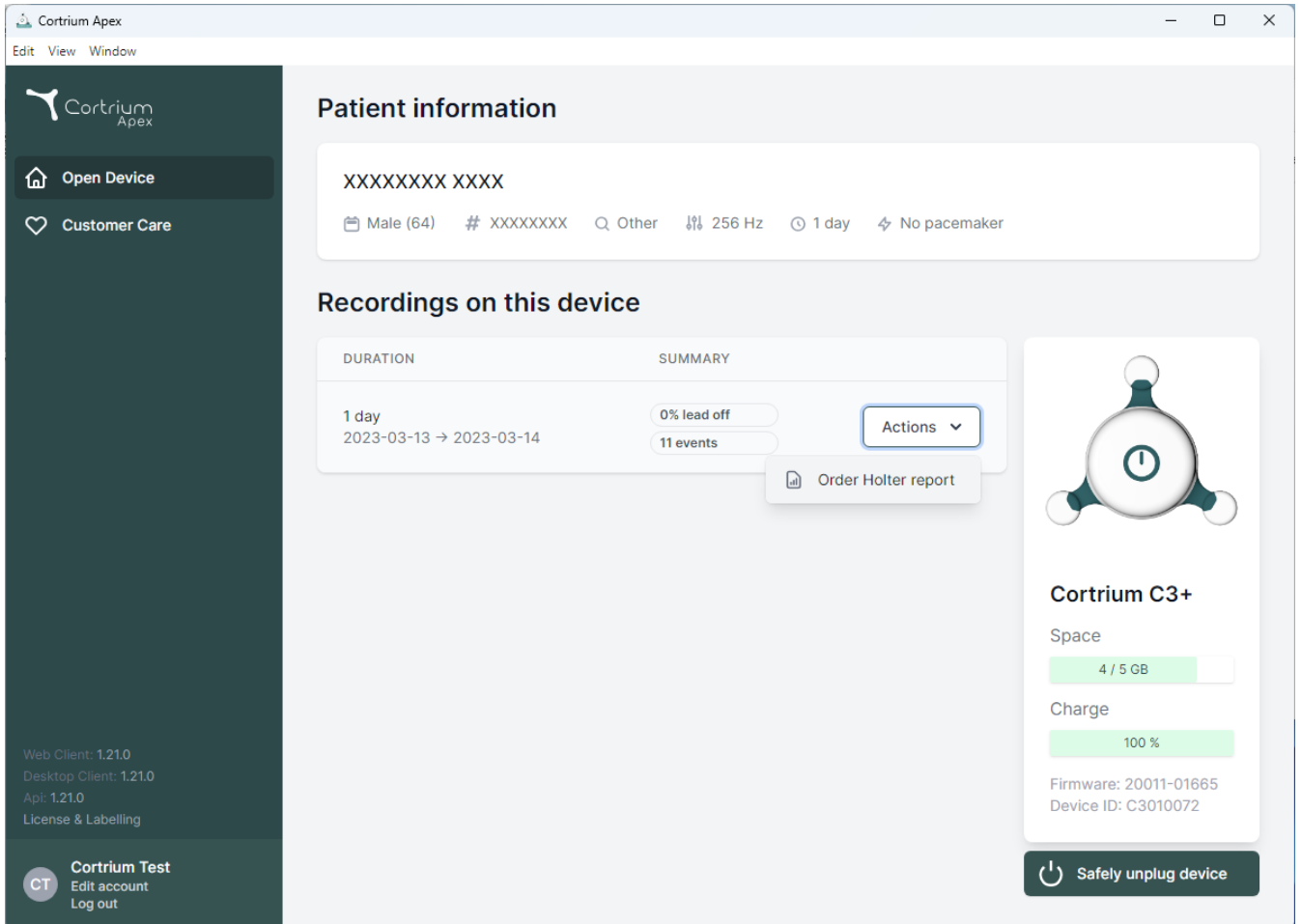The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo, 'Open Device', and 'Customer Care' links. The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

Actions

Order Holter report

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

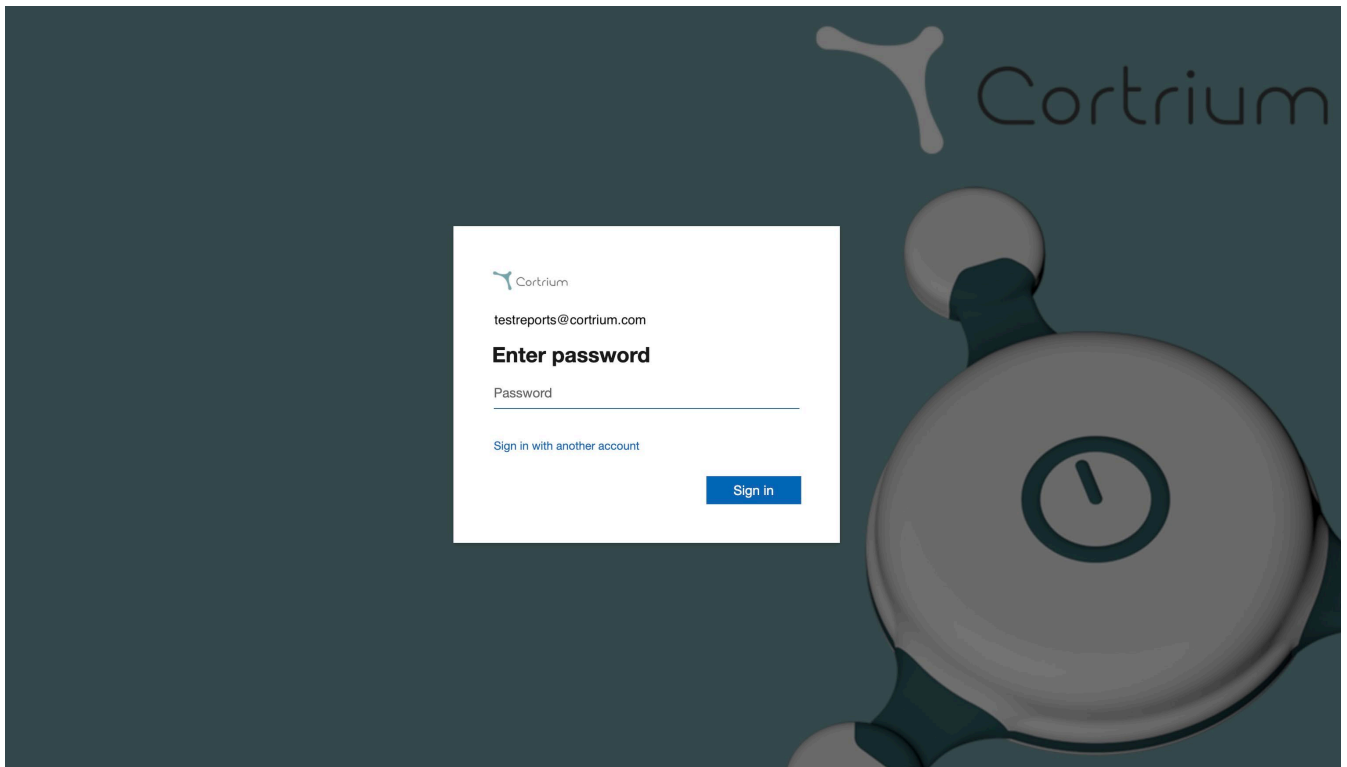
Safely unplug device

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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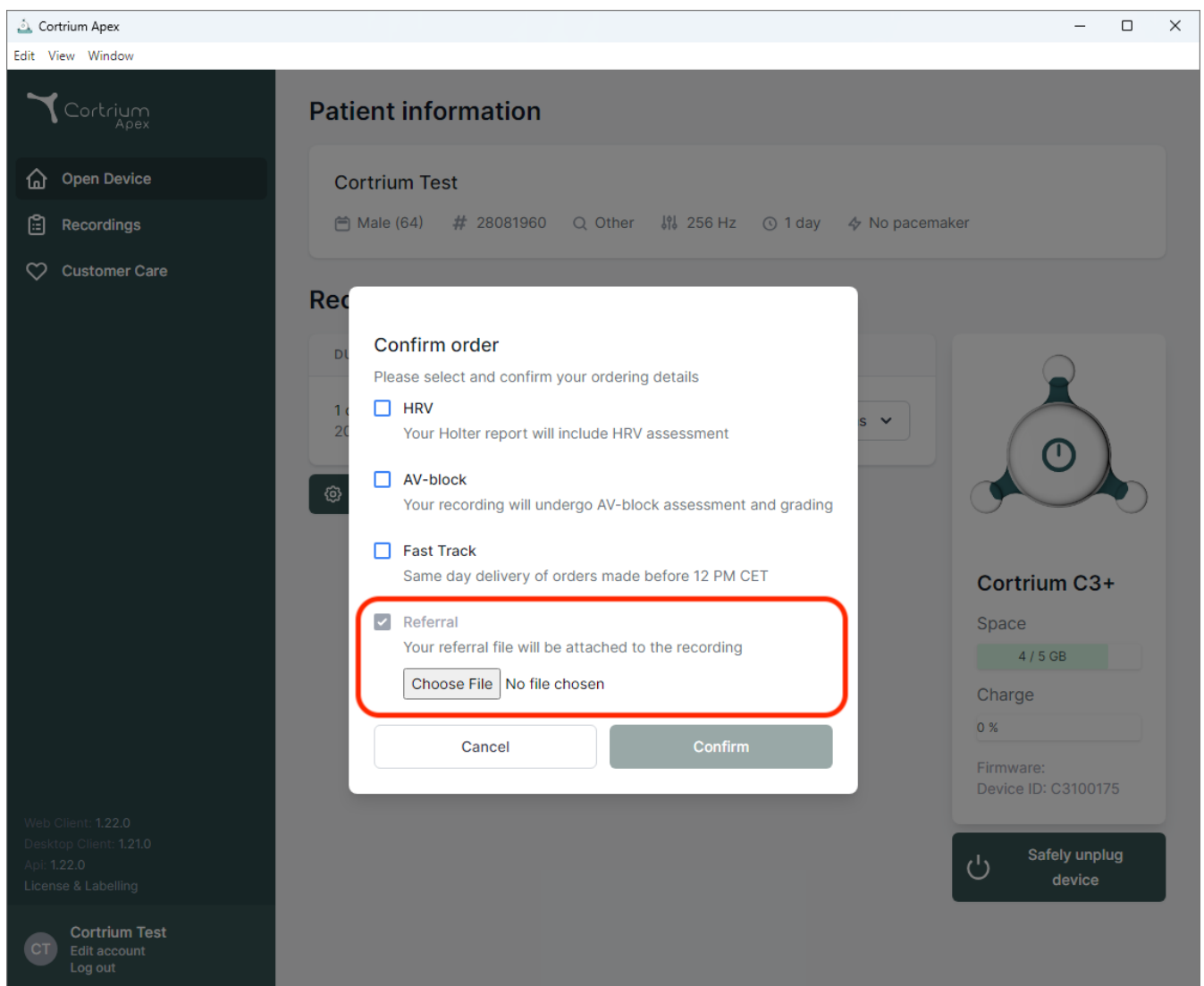
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
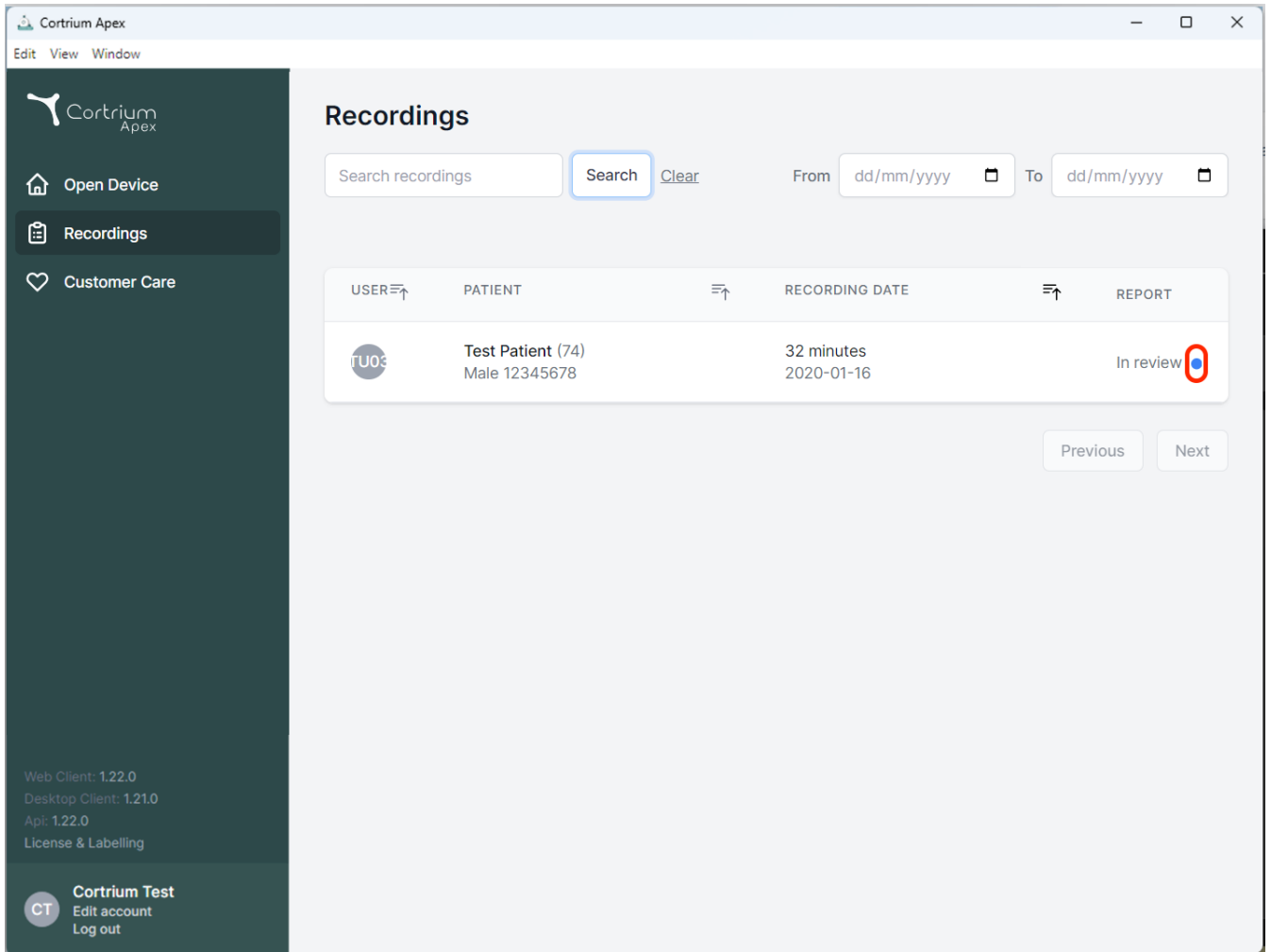
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation options: "Open Device", "Recordings" (selected), and "Customer Care". The main area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. A single recording is listed for user "TU03", patient "Test Patient (74) Male 12345678", with a duration of "32 minutes" and a date of "2020-01-16". The status "In review" is shown next to a blue dot icon, which is circled in red. At the bottom right of the table are "Previous" and "Next" buttons. The footer of the sidebar shows version information: "Web Client: 1.22.0", "Desktop Client: 1.21.0", "Api: 1.22.0", and "License & Labelling". The user "Cortrium Test" is logged in, with options to "Edit account" and "Log out".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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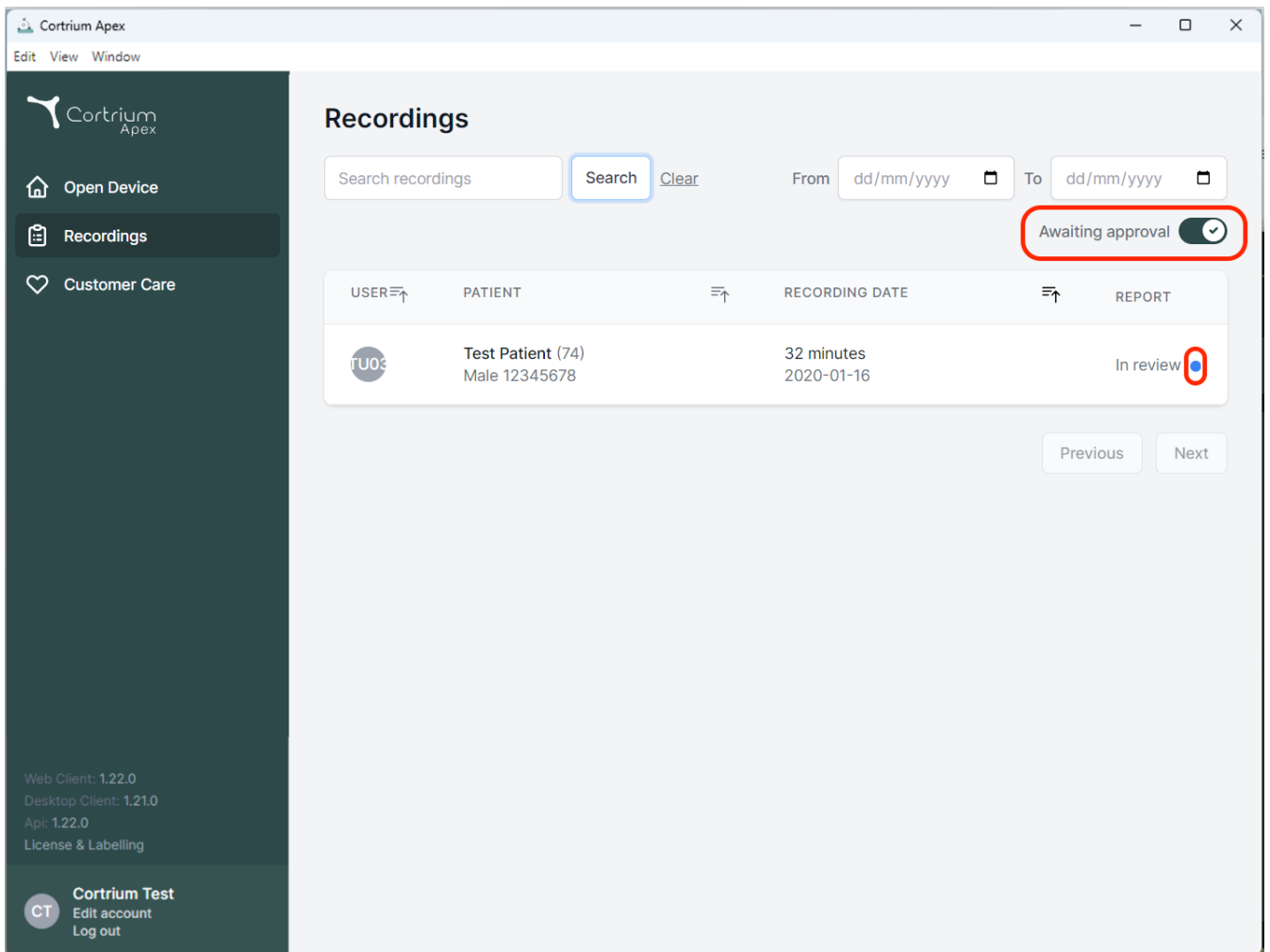
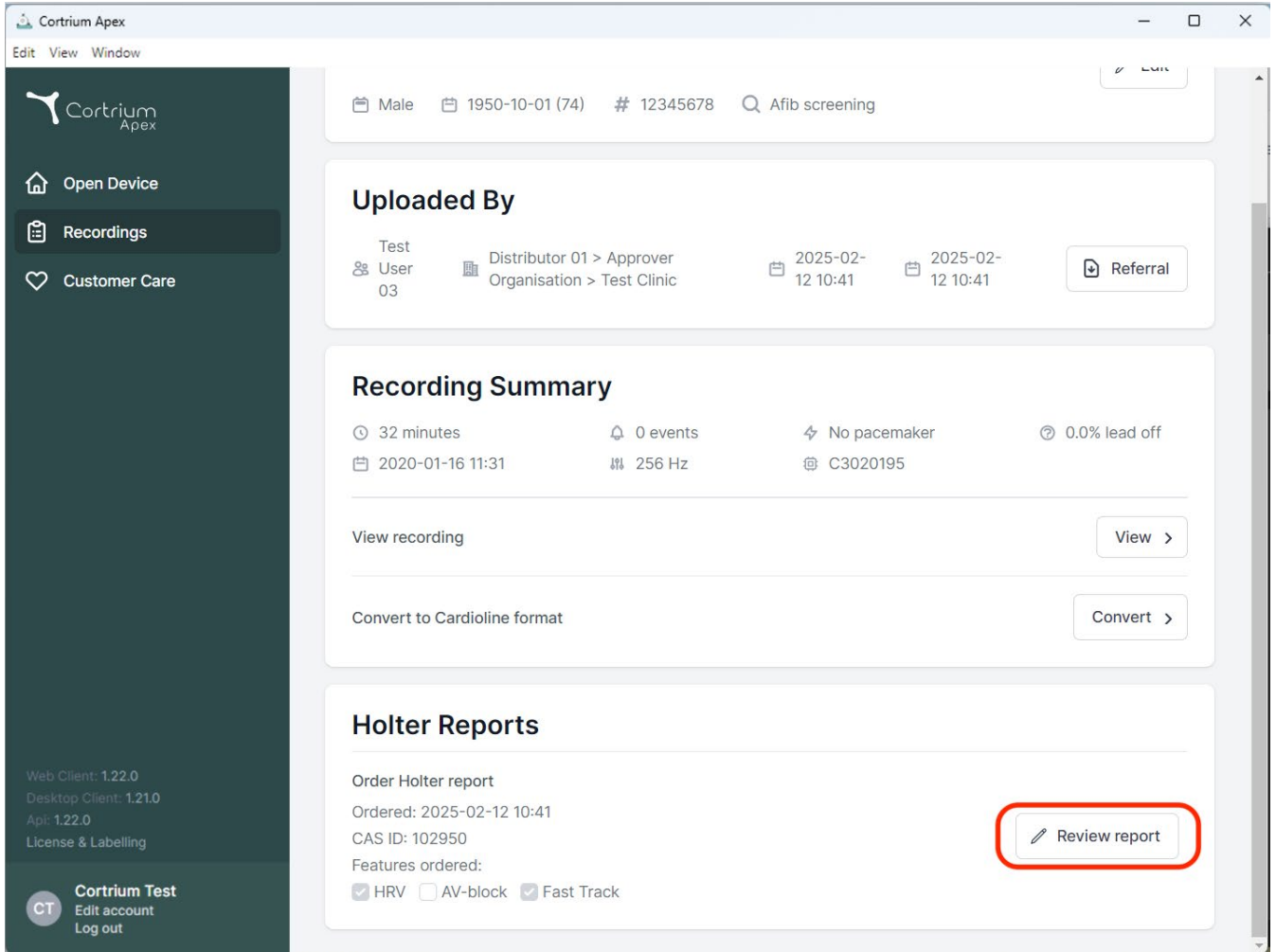
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section lists 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with timestamps of 2025-02-12 10:41 and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It also shows a frequency of 256 Hz and a device ID C3020195. Options to 'View recording' and 'Convert to Cardioline format' are available. The 'Holter Reports' section includes an 'Order Holter report' button, an 'Ordered' date of 2025-02-12 10:41, a CAS ID of 102950, and a list of ordered features: HRV (checked), AV-block (unchecked), and Fast Track (checked). A 'Review report' button is highlighted with a red rectangle.

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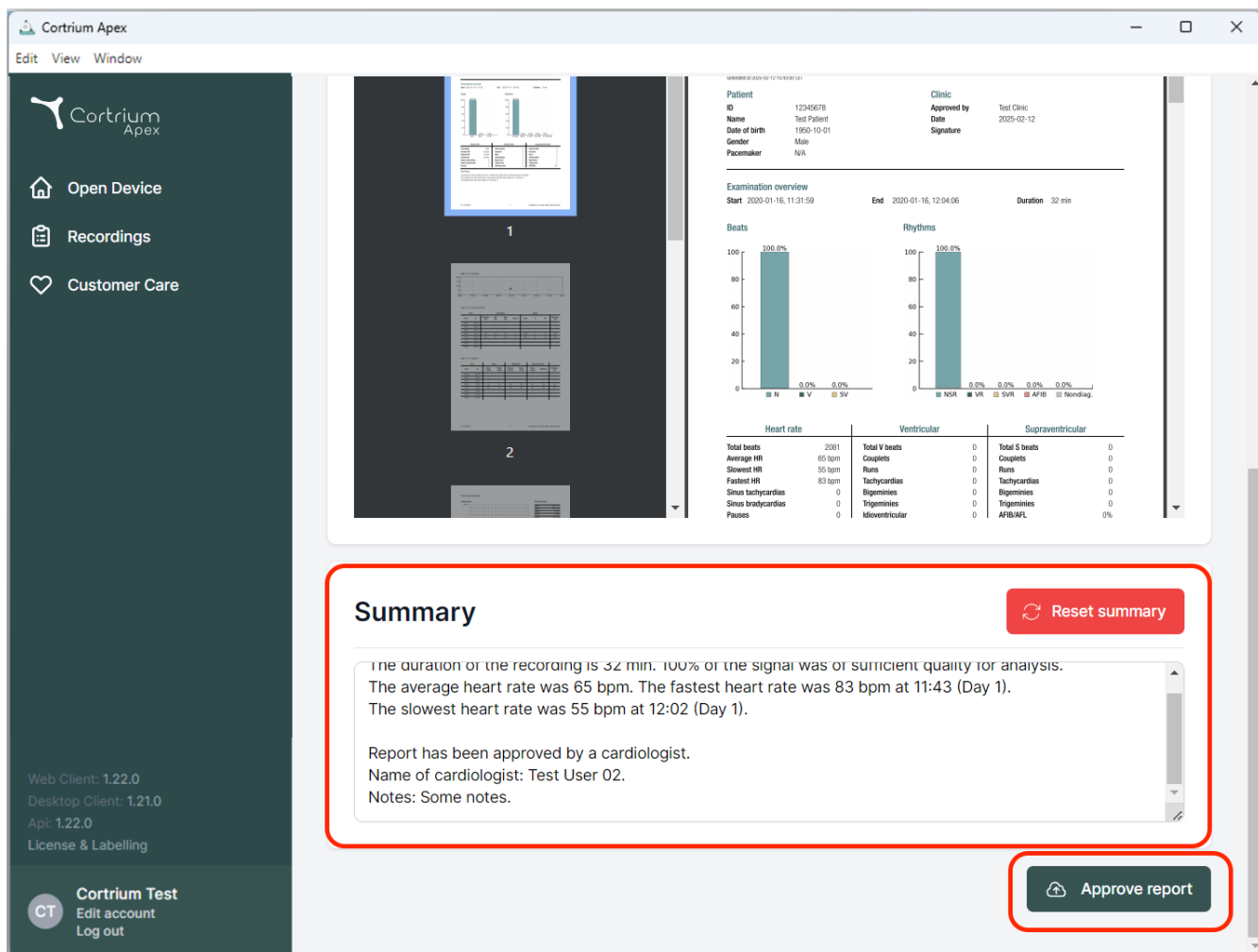Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient
ID: 12345678
Name: Test Patient
Date of birth: 1950-10-01
Gender: Male
Pacemaker: N/A

Clinic
Approved by: Test Clinic
Date: 2025-02-12
Signature:

Examination overview
Start: 2020-01-16, 11:31:59
End: 2020-01-16, 12:04:06
Duration: 32 min

Beats
100.0%
0.0%
0.0%

Rhythms
100.0%
0.0%
0.0%
0.0%
0.0%

Heart rate		Ventricular		Supraventricular	
Total beats	2081	Total V beats	0	Total S beats	0
Average HR	65 bpm	Couplets	0	Couplets	0
Slowest HR	55 bpm	Runs	0	Runs	0
Fastest HR	83 bpm	Tachycardias	0	Tachycardias	0
Sinus tachycardia	0	Bigeminy	0	Bigeminy	0
Sinus bradycardia	0	Trigeminy	0	Trigeminy	0
Pauses	0	Idioventricular	0	AFIB/AFL	0%

Summary Reset summary

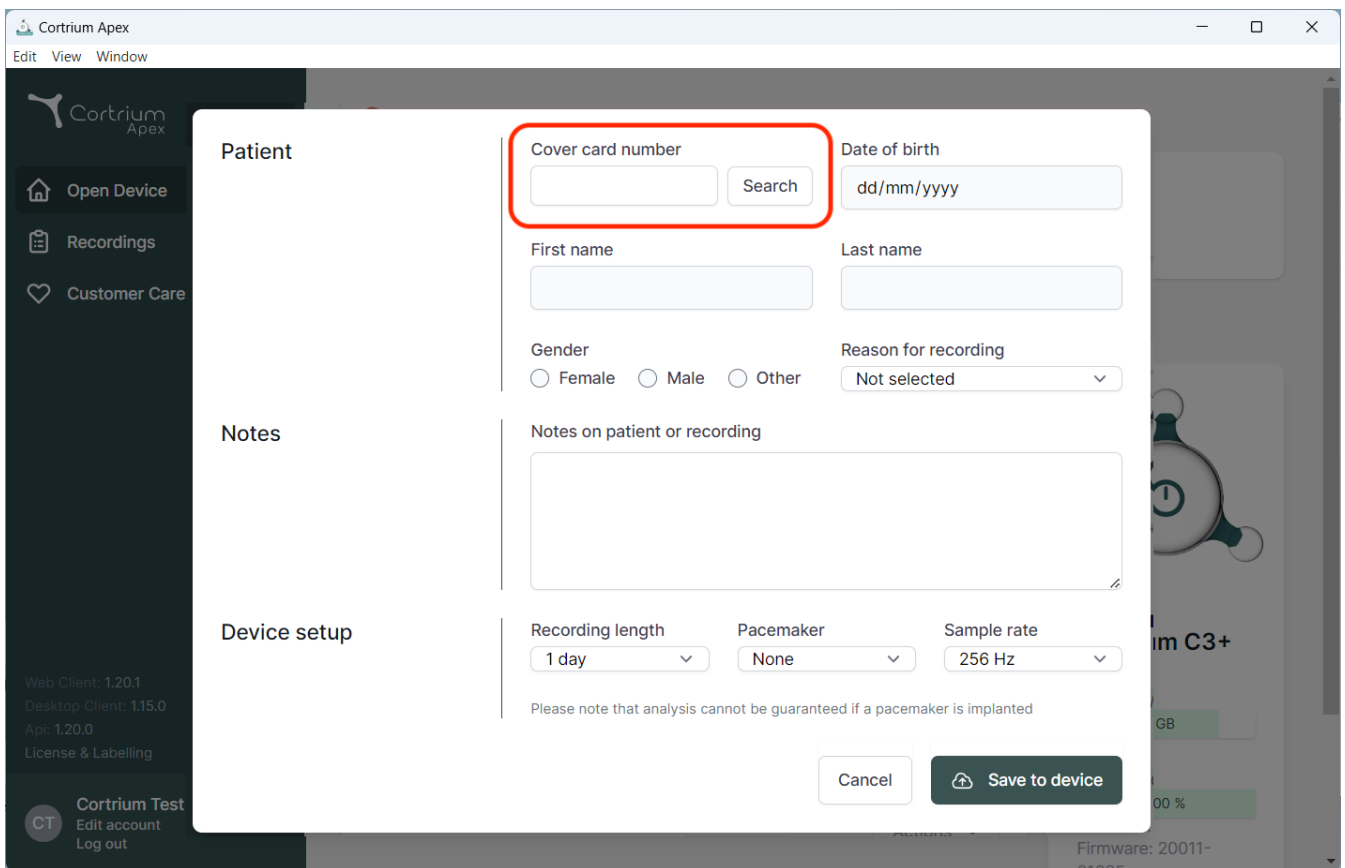
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot displays the Cortrium Apex software interface. A modal window titled "Patient" is open, allowing for patient search and recording setup. The form includes the following fields and options:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes on patient or recording:** A large text area for additional notes.
- Device setup:**
 - Recording length:** A dropdown menu showing "1 day".
 - Pacemaker:** A dropdown menu showing "None".
 - Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup options, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

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Cortrium Apex toetab järgnevaid EKG salvesteid

- Cortrium C3w Holteri monitor
- Cortrium C3+ Holteri monitor
- Livetec kECG-3

05.2026

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Taani
E-post: info@cortrium.com • Veebisait: www.cortrium.com

Cortrium Apex UDI(01)05745000379033