# Cortrium Apex 1.27

# Cortrium Apex 1.27

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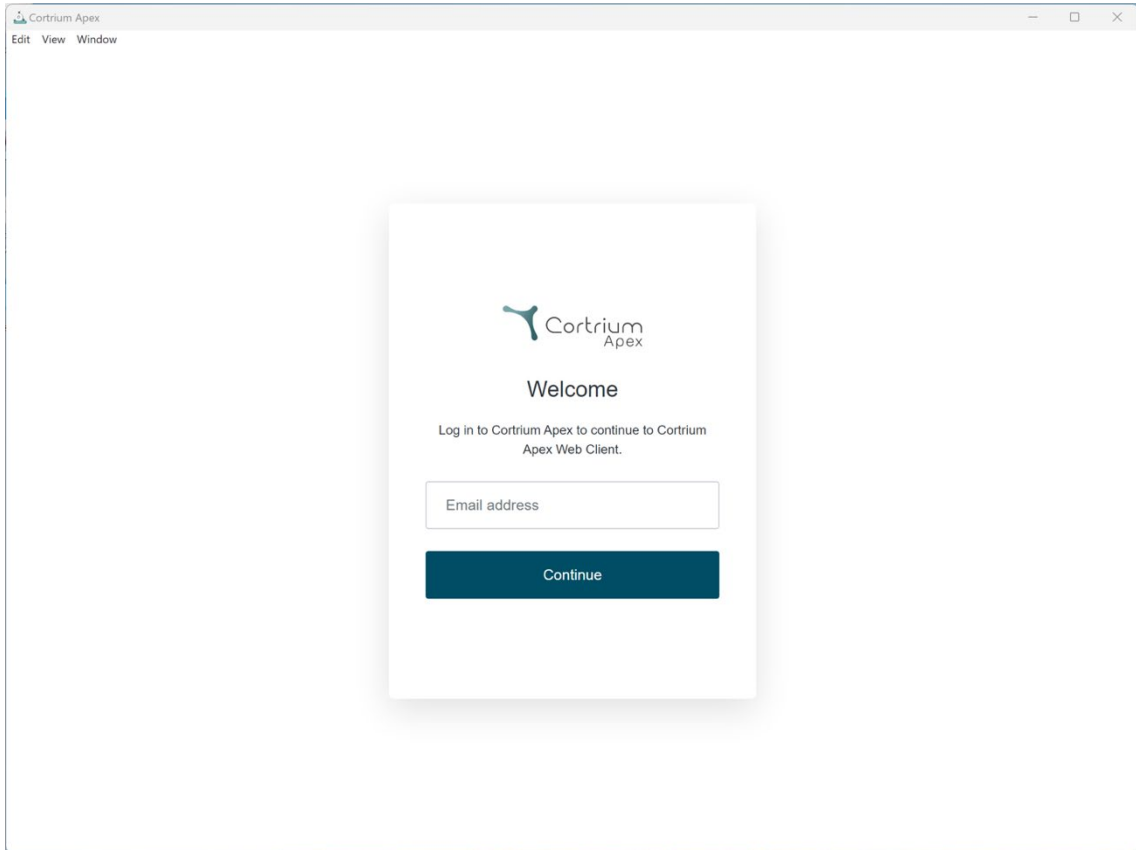
 Cortrium  
Apex

Welcome


Log in to Cortrium Apex to continue to Cortrium  
Apex Web Client.

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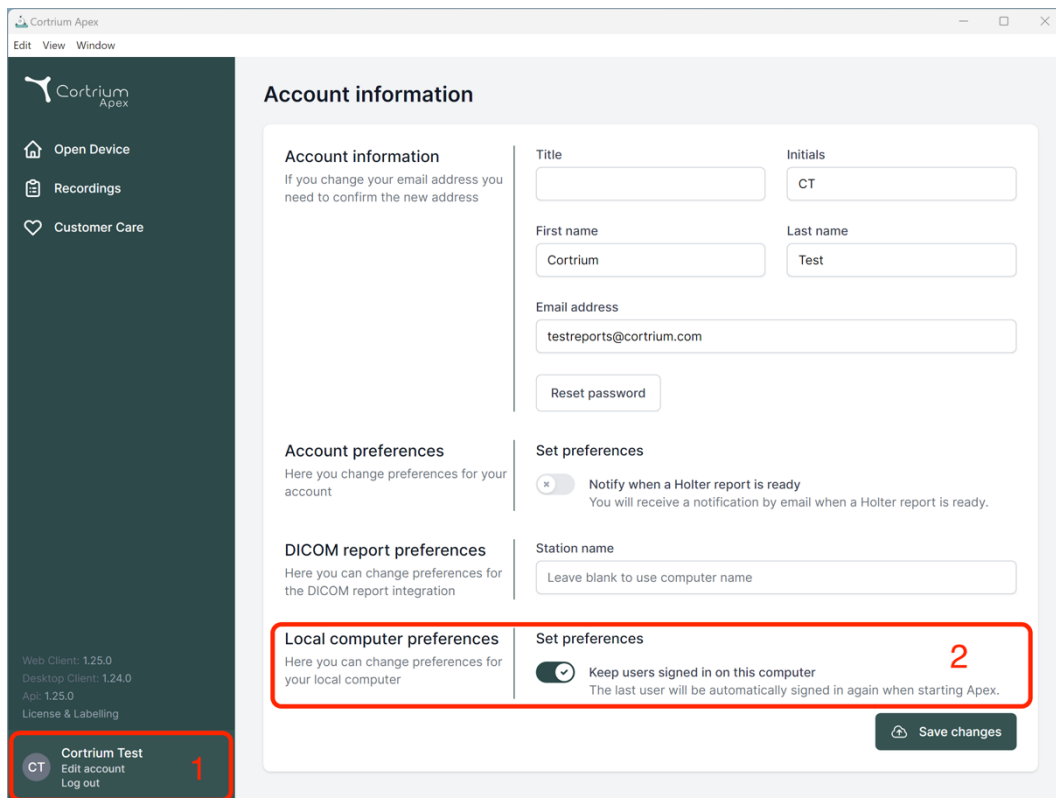The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, the user's name 'Cortrium Test' is displayed with a red box and a red number '1' next to it, along with links for 'Edit account' and 'Log out'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' with a toggle for 'Notify when a Holter report is ready'; 'DICOM report preferences' with a field for 'Station name'; and 'Local computer preferences' with a toggle for 'Keep users signed in on this computer' (marked with a red box and a red number '2'). A 'Save changes' button is located at the bottom right of the main content area.

**Account information**

If you change your email address you need to confirm the new address

Title:  Initials:

First name:  Last name:

Email address:

**Account preferences**

Here you change preferences for your account

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**DICOM report preferences**

Here you can change preferences for the DICOM report integration

Station name:

**Local computer preferences**

Here you can change preferences for your local computer

☒ **Keep users signed in on this computer**  
The last user will be automatically signed in again when starting Apex.

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
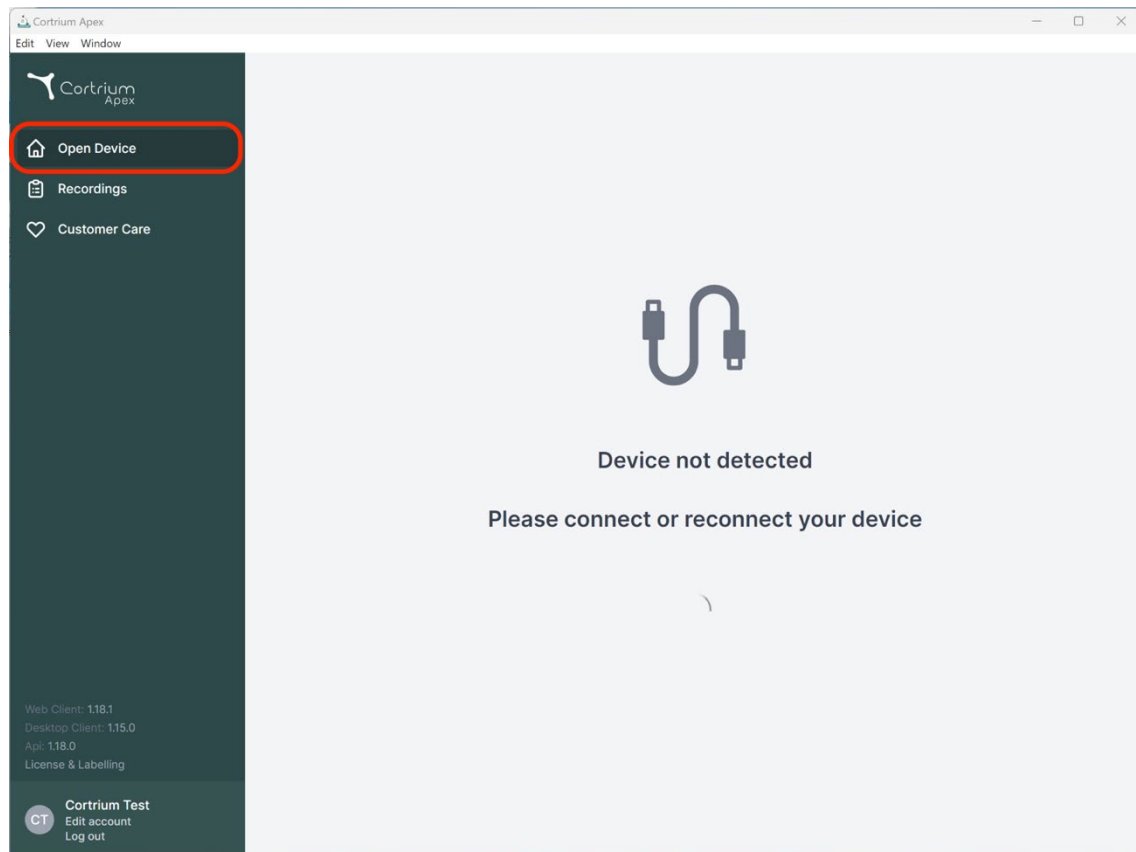
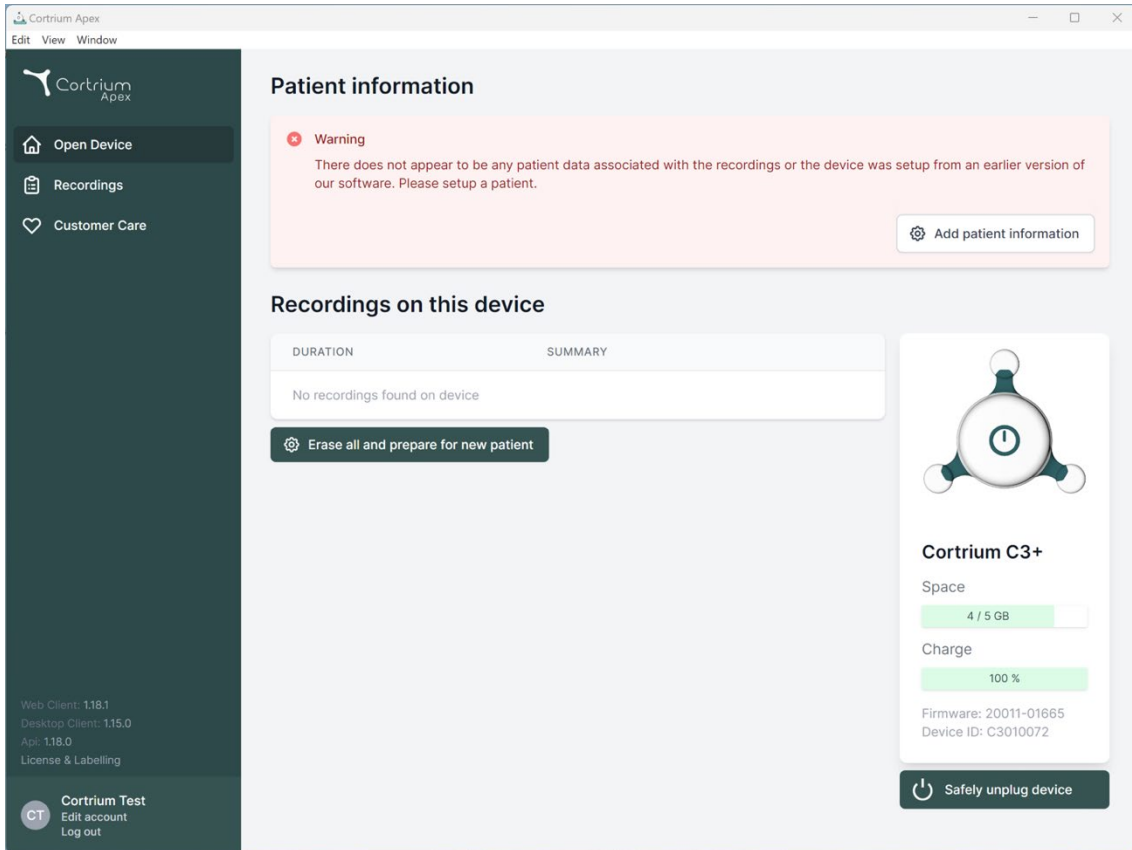
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, it lists version information: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'Api: 1.18.0', and 'License & Labelling'. Below this is a 'Cortrium Test' section with 'Edit account' and 'Log out' options.

The main content area has a light gray background. At the top, under the heading 'Patient information', there is a pink warning box with a red 'x' icon. The text inside the warning box reads: 'Warning. There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' To the right of the warning box is a button labeled 'Add patient information' with a gear icon.

Below the warning box is the heading 'Recordings on this device'. Under this heading is a table with two columns: 'DURATION' and 'SUMMARY'. The table is currently empty, with the text 'No recordings found on device' centered below the column headers. Below the table is a dark button labeled 'Erase all and prepare for new patient' with a gear icon.

On the right side of the main content area is a white card for the 'Cortrium C3+' device. At the top of the card is an illustration of the device. Below the illustration, the card displays the following information: 'Space' with a progress bar showing '4 / 5 GB', 'Charge' with a progress bar showing '100 %', 'Firmware: 20011-01665', and 'Device ID: C3010072'. At the bottom of the card is a dark button labeled 'Safely unplug device' with a power icon.

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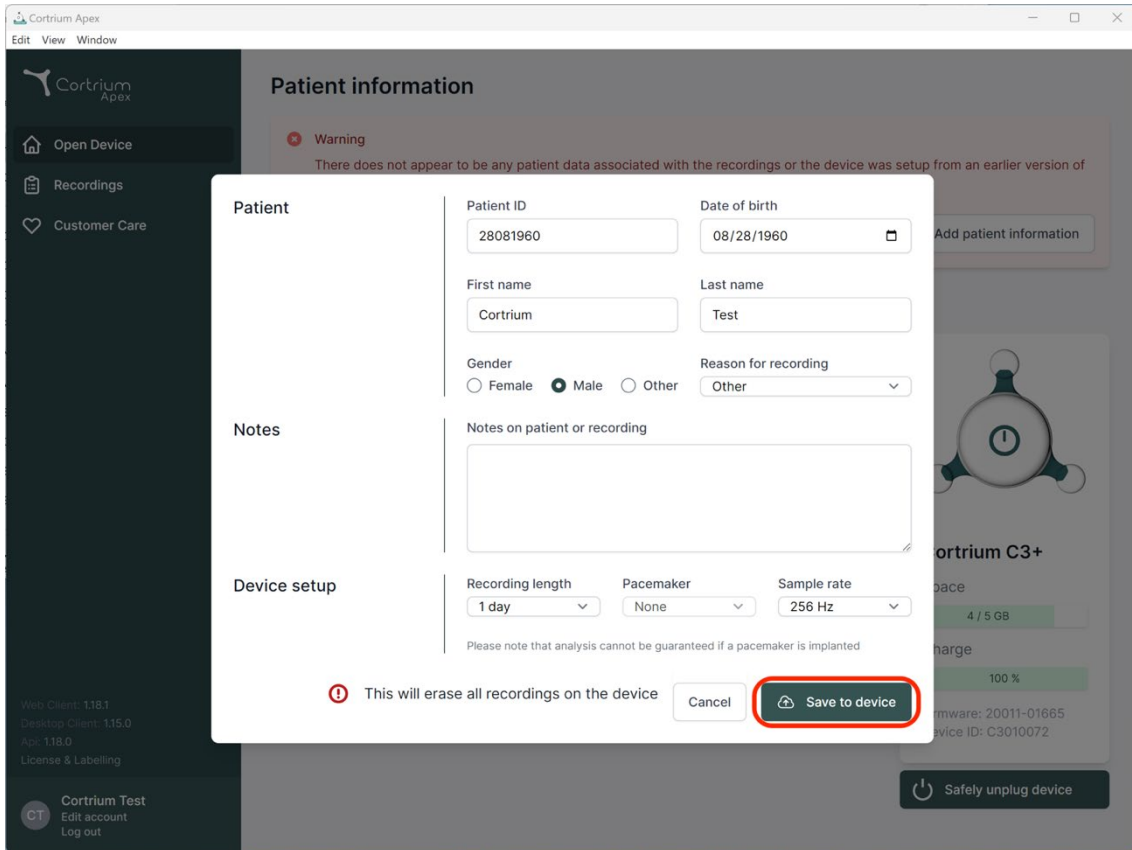**Patient information**

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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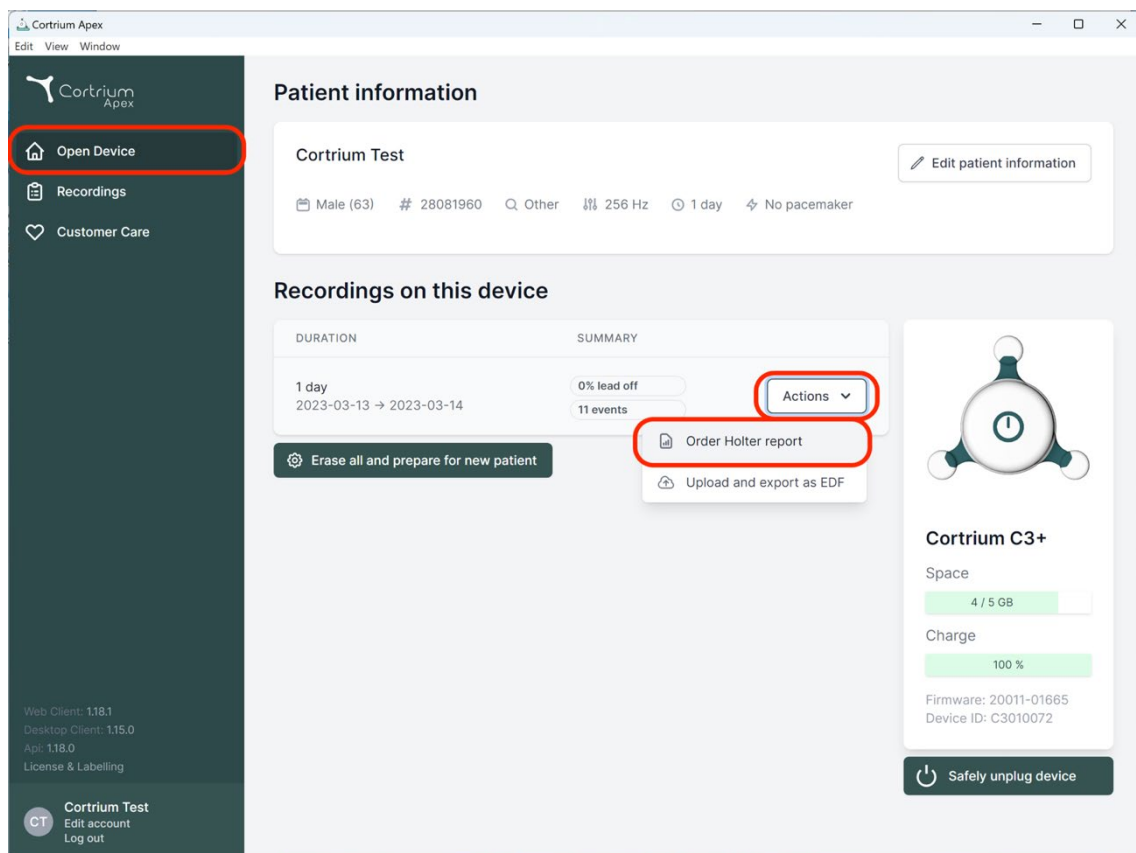
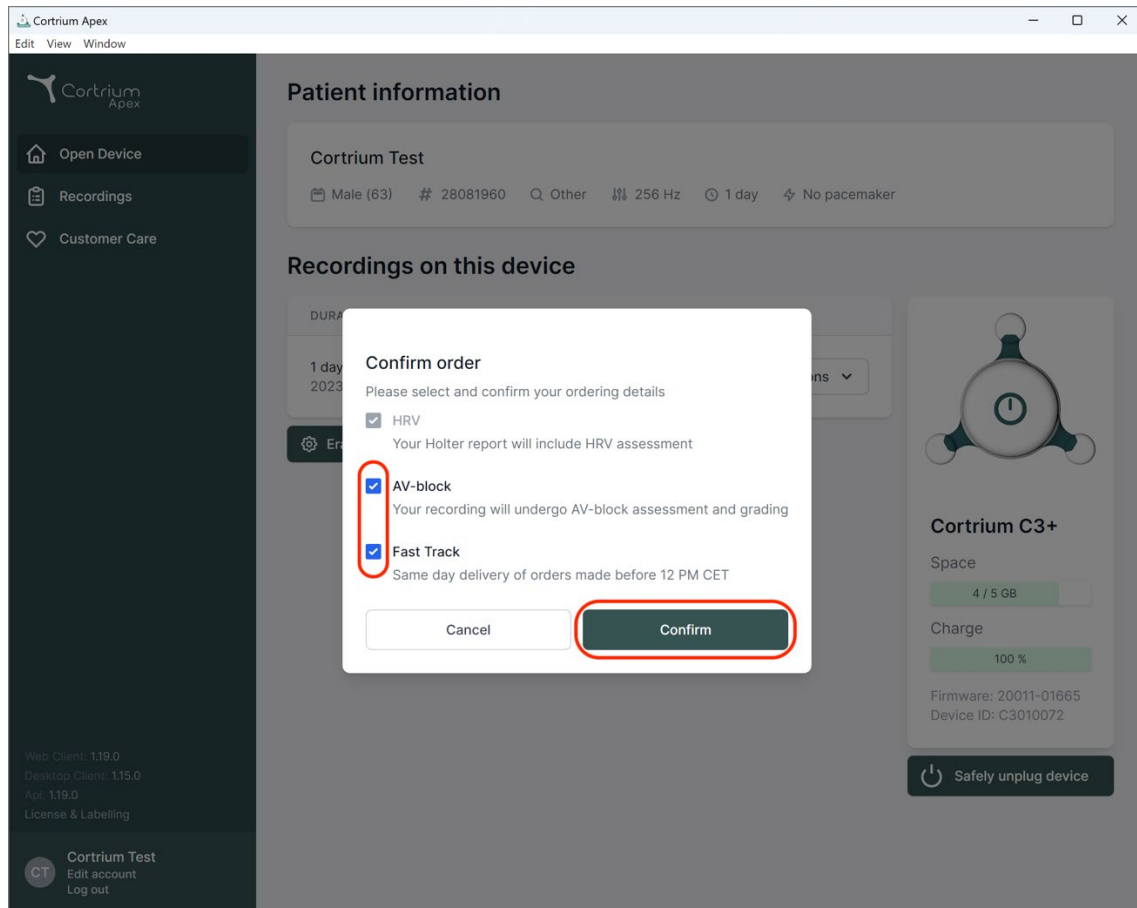
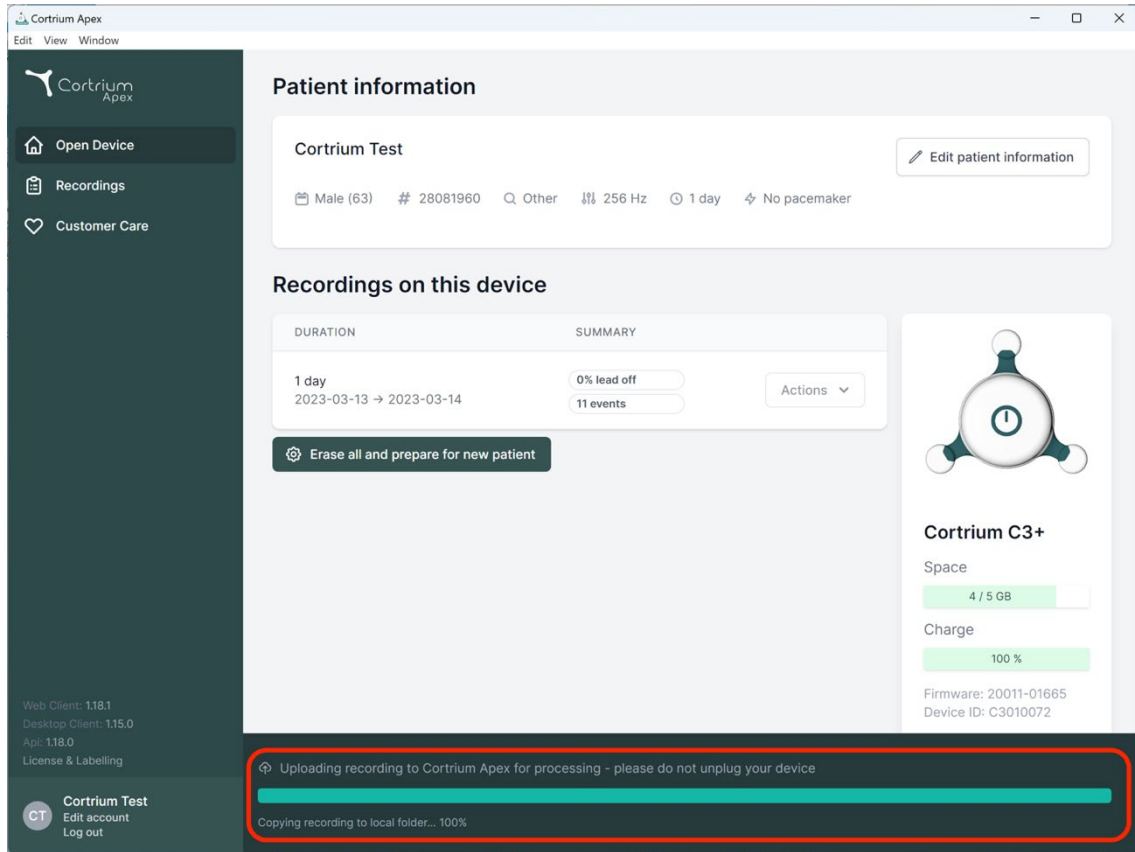
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including patient demographics and device settings. Below this, the 'Recordings on this device' section shows a table with columns for DURATION and SUMMARY. A recording for '1 day' (2023-03-13 to 2023-03-14) is listed, showing 0% lead off and 11 events. A button 'Erase all and prepare for new patient' is visible. On the right, a device status card for 'Cortrium C3+' shows space (4 / 5 GB) and charge (100 %) levels, along with firmware and device ID information. At the bottom, a red-bordered box highlights a status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and a progress bar for 'Copying recording to local folder... 100%'.

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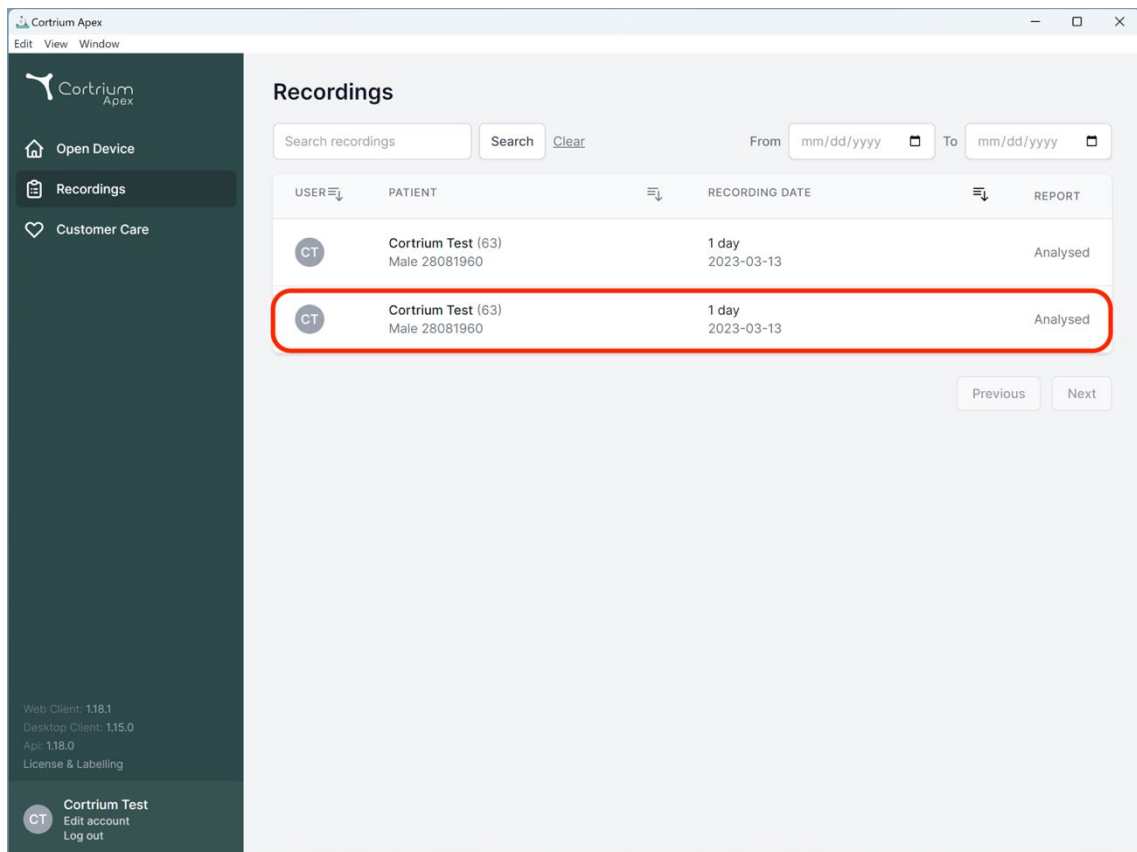
**Recordings**

Search recordings   Search   Clear   From mm/dd/yyyy   To mm/dd/yyyy

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
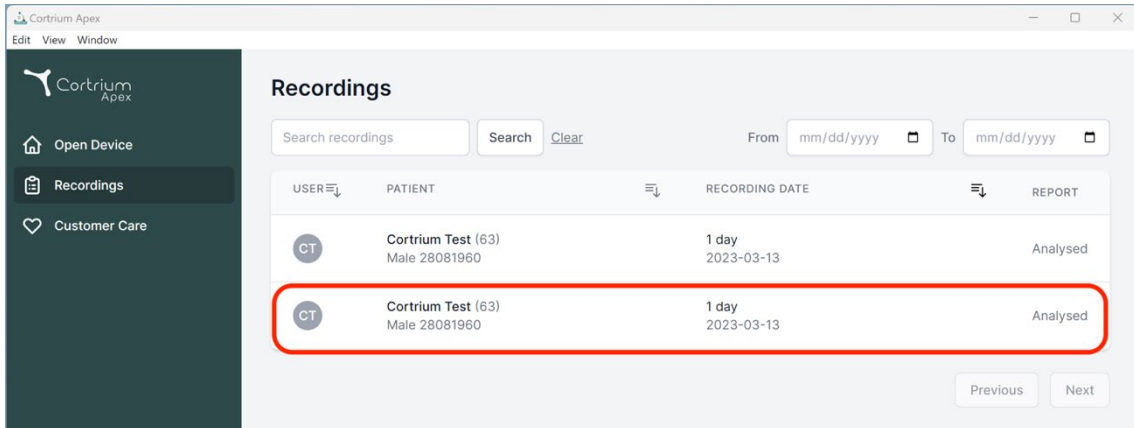
The screenshot shows the 'Recordings' page in the Cortrium Apex application. The left sidebar contains navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main area displays a table of recordings with columns: USER, PATIENT, RECORDING DATE, and REPORT. The second row is highlighted with a red border.

| USER | PATIENT                             | RECORDING DATE      | REPORT   |
|------|-------------------------------------|---------------------|----------|
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |

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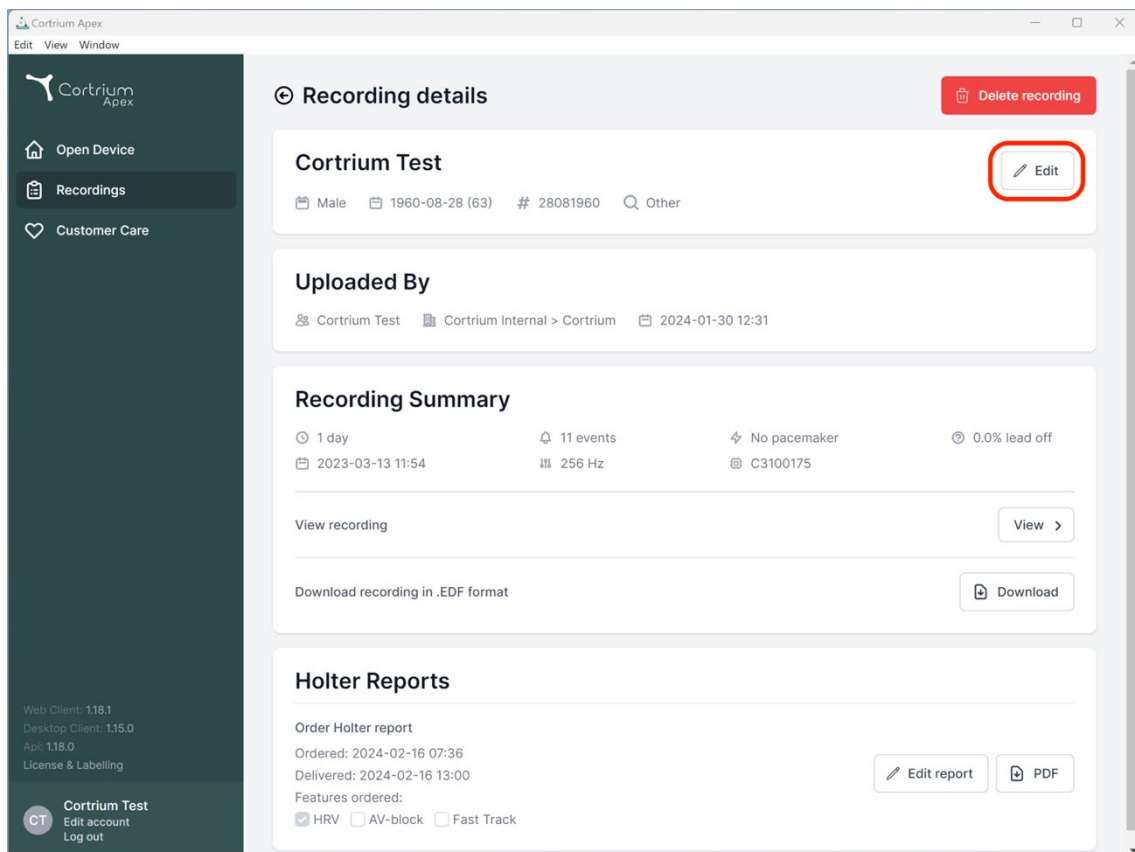The screenshot shows the 'Recording details' page in the Cortrium Apex application. The left sidebar contains navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main area displays the details for a recording titled 'Cortrium Test'. The 'Edit' button is highlighted with a red border.

**Recording details**

**Cortrium Test**

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

**View recording** View >

**Download recording in .EDF format** Download

**Holter Reports**

Order Holter report

Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
☒ HRV ☐ AV-block ☐ Fast Track

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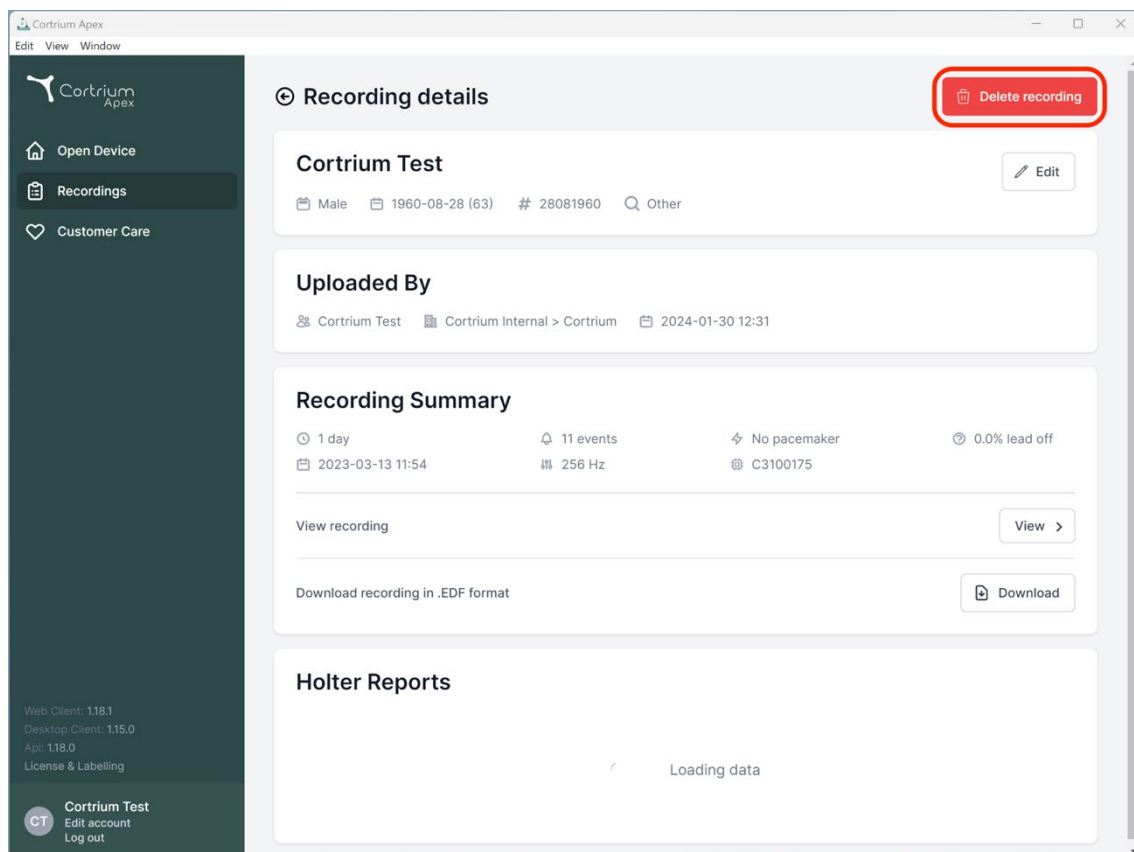
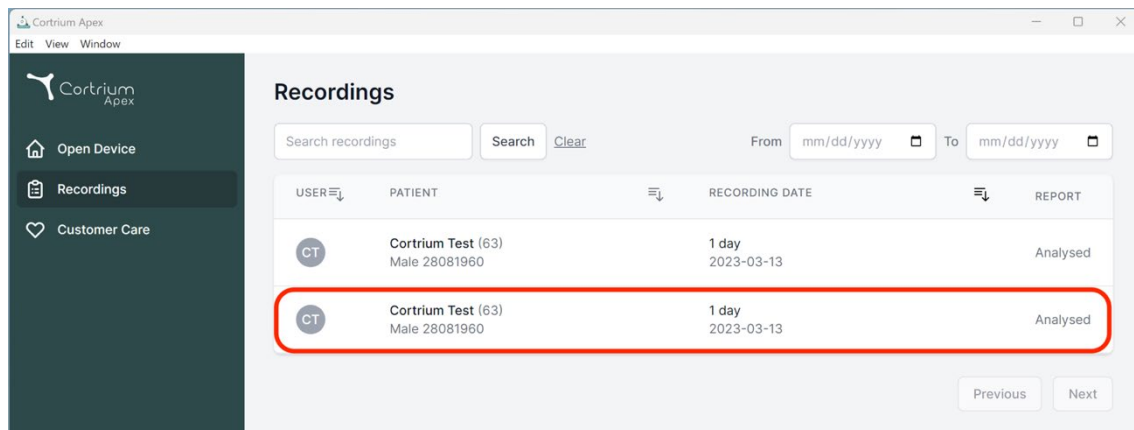
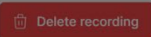
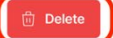
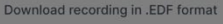
### Uploaded By

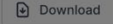
 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

**Confirm delete**  
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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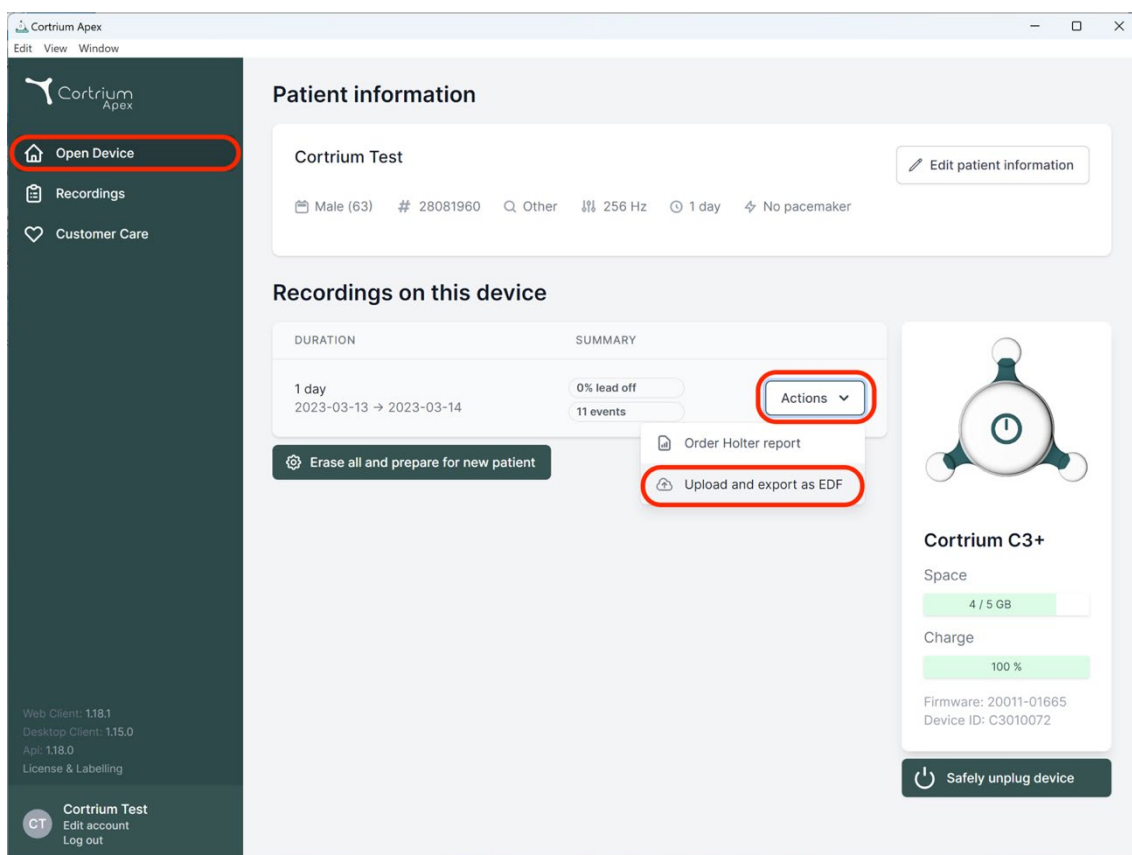
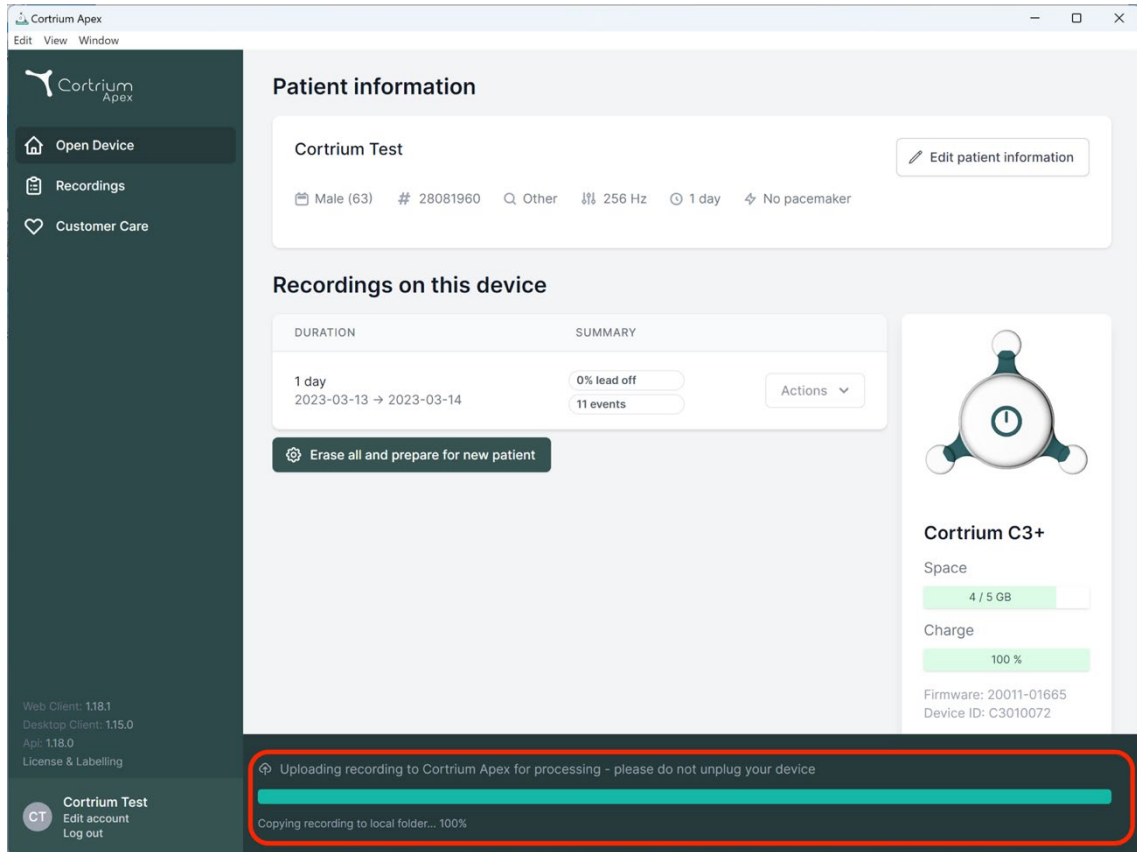
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  |                         |
|----------------------------------|--------------------------|-------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events | <a href="#">Actions</a> |

[Erase all and prepare for new patient](#)

**Cortrium C3+**

Space  
4 / 5 GB

Charge  
100 %

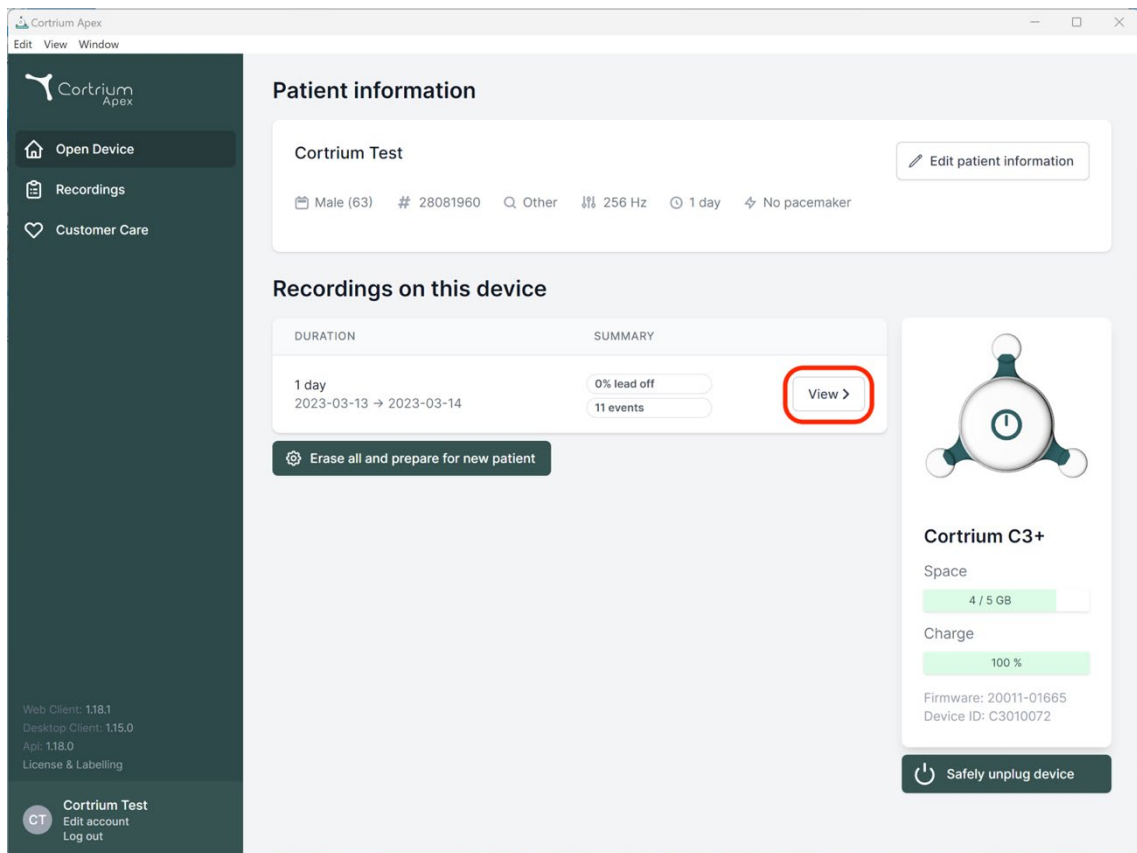
Firmware: 20011-01665  
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

**Cortrium Test**  
CT Edit account Log out

**Cortrium Apex**  
Edit View Window

**Patient information**

**Cortrium Test** [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  |                           |
|----------------------------------|--------------------------|---------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events | <a href="#">View &gt;</a> |

[Erase all and prepare for new patient](#)

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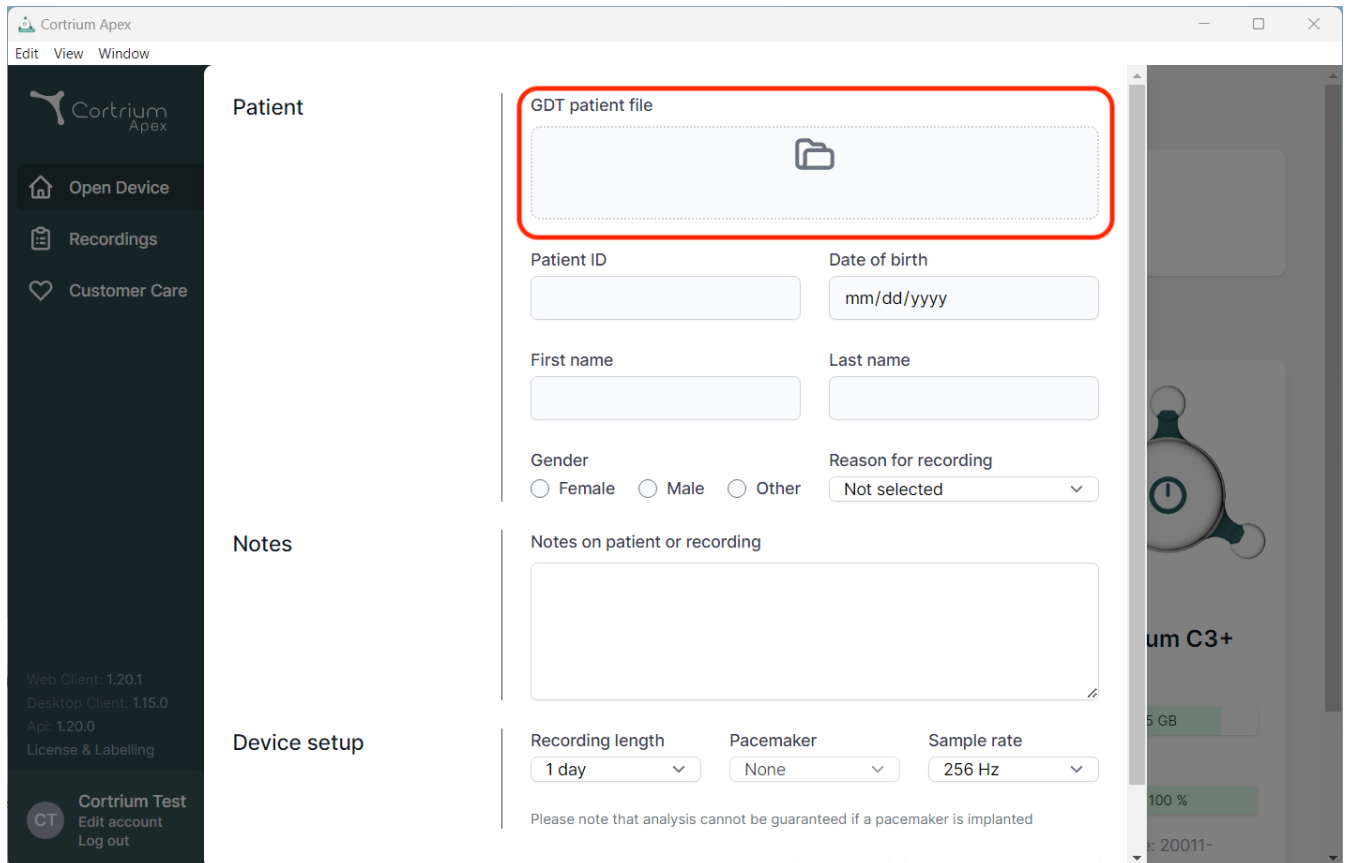
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section is highlighted and contains a form for entering patient information. The form includes a red-bordered box for the GDT patient file, followed by input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, and Last name. Below these are radio buttons for Gender (Female, Male, Other) and a dropdown menu for Reason for recording (Not selected). The Notes section has a text area for notes on the patient or recording. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

**Patient**

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

**Notes**

Notes on patient or recording

**Device setup**

Recording length

1 day

Pacemaker

None

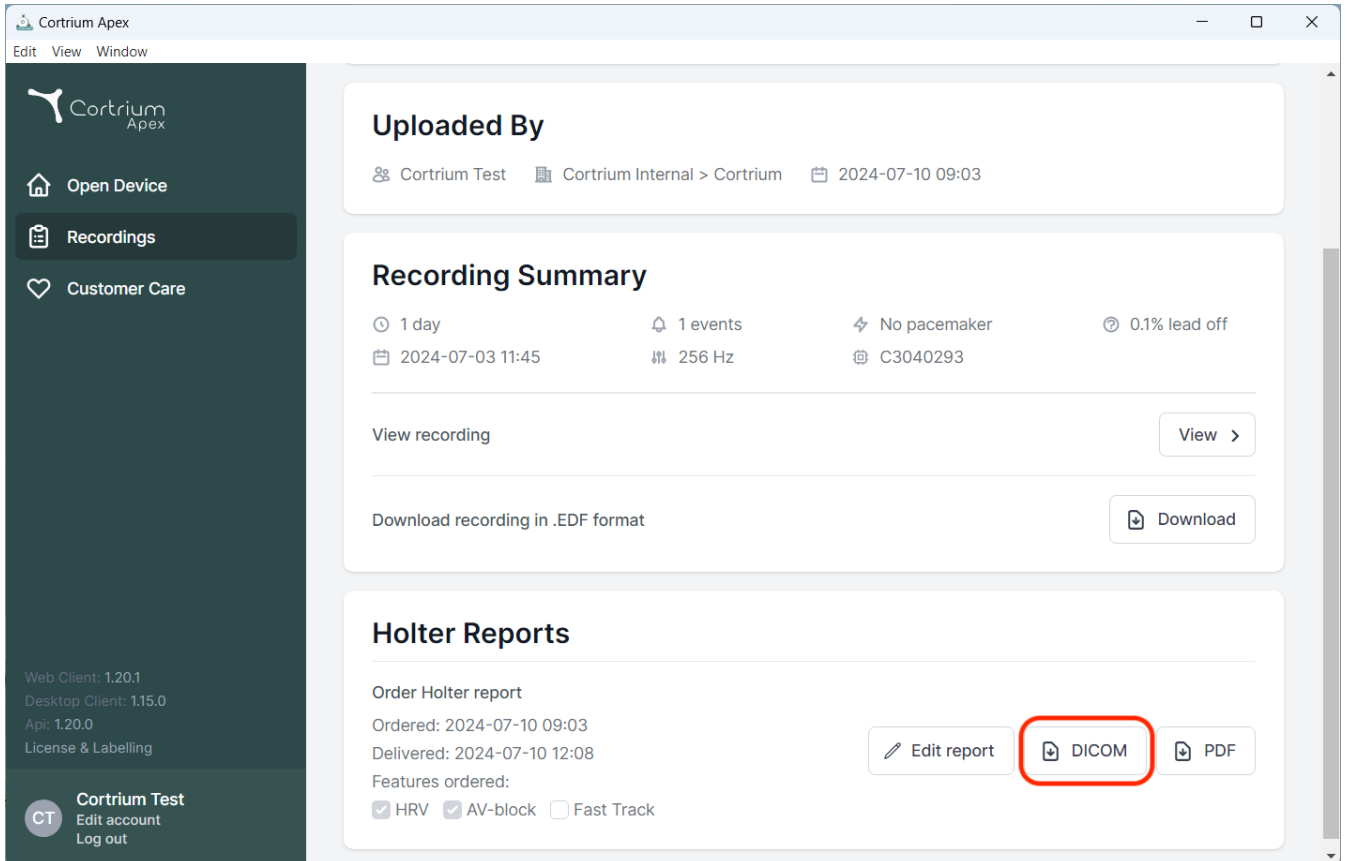
Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (labeled 'Download recording in .EDF format').
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

The bottom of the sidebar contains version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a user profile for 'Cortrium Test' with 'Edit account' and 'Log out' options.

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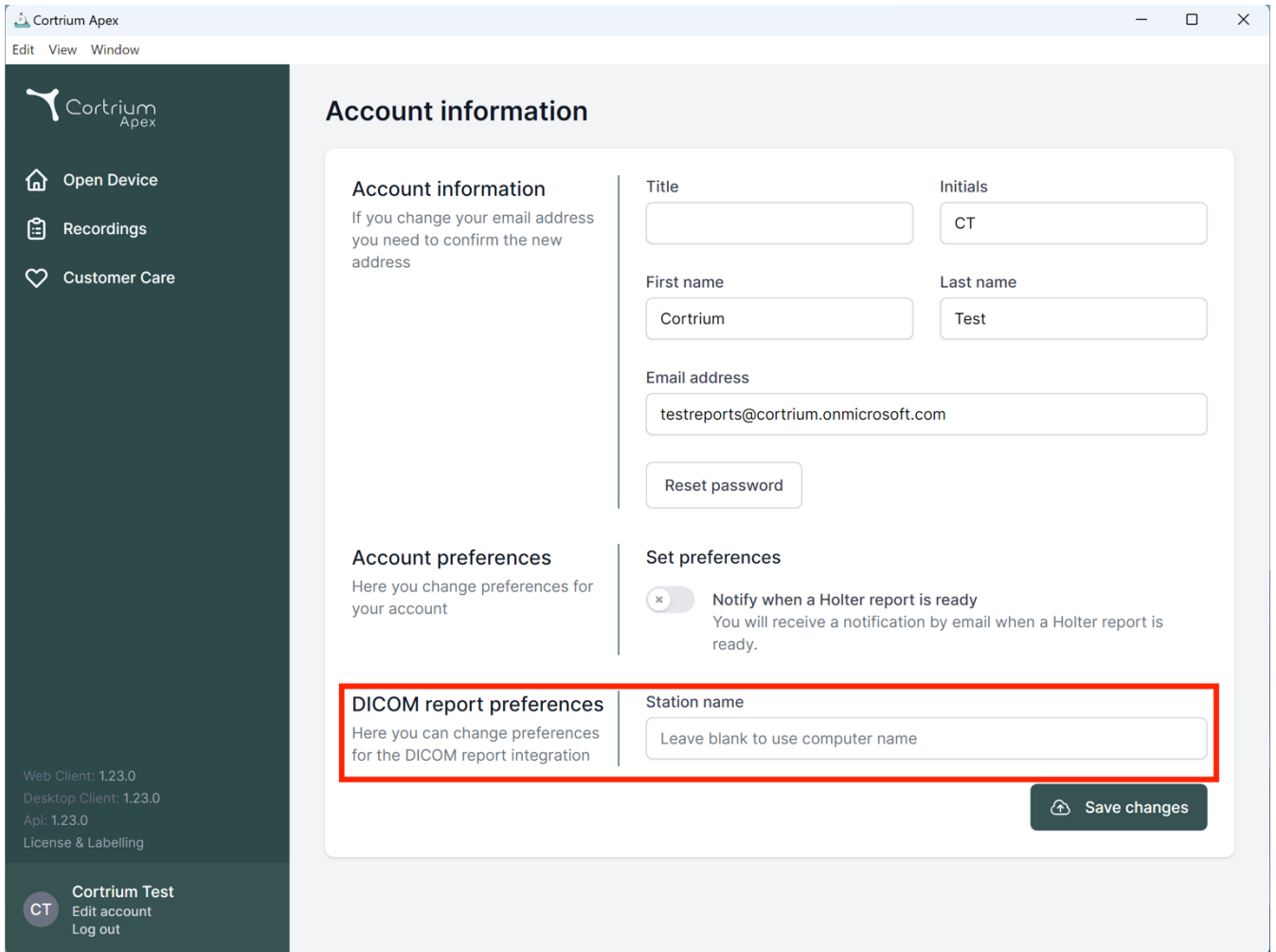**Cortrium Apex**  
Edit View Window

**Account information**

**Account information**  
If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

**Account preferences**  
Here you change preferences for your account

**Set preferences**

☒ **Notify when a Holter report is ready**  
You will receive a notification by email when a Holter report is ready.

**DICOM report preferences**  
Here you can change preferences for the DICOM report integration

Station name:

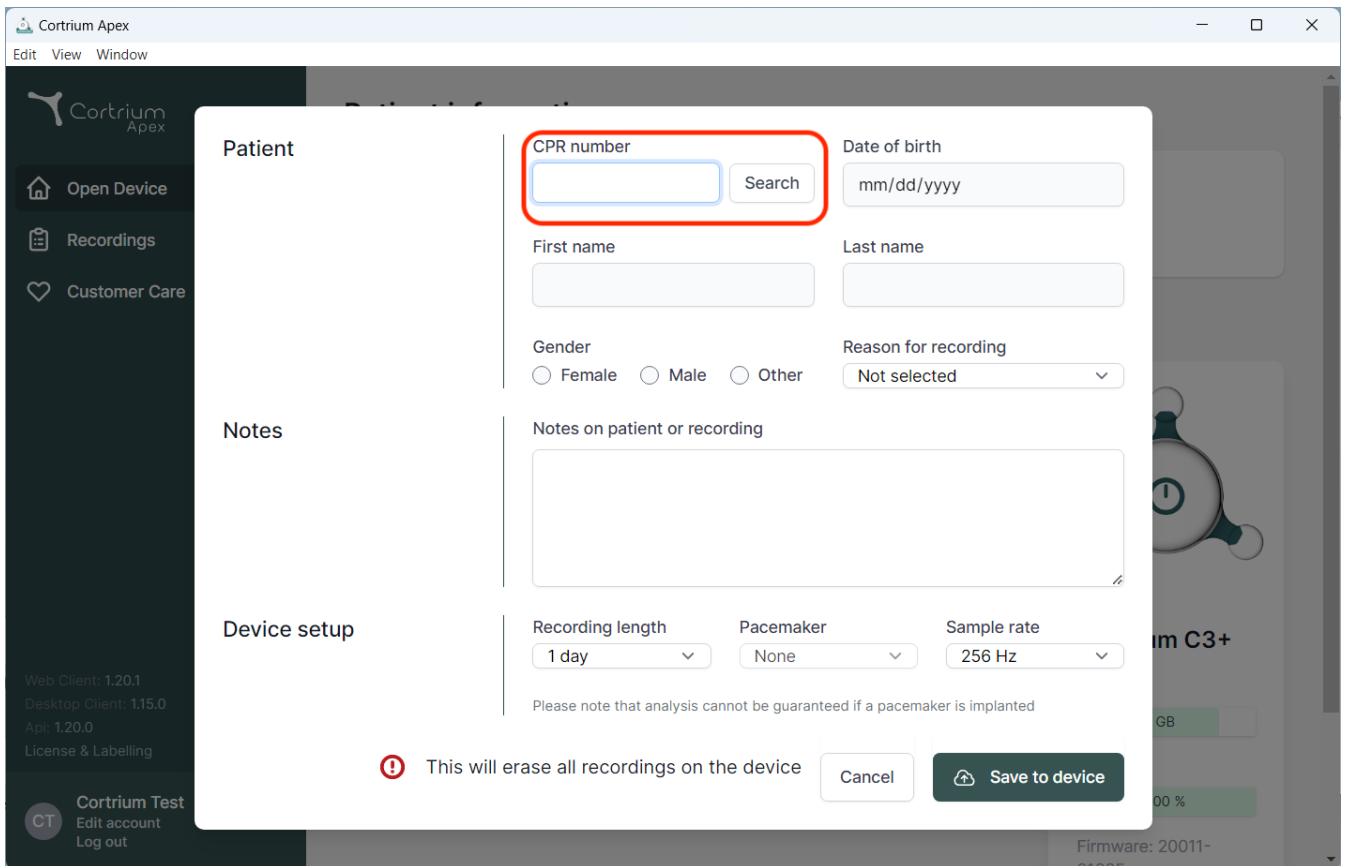
[Save changes](#)

Web Client: 1.23.0  
Desktop Client: 1.23.0  
Api: 1.23.0  
License & Labelling

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The screenshot shows the Cortrium Apex web application interface. A modal window is open for patient search. The modal has a sidebar on the left with 'Patient', 'Notes', and 'Device setup' sections. The main area contains search fields: 'CPR number' (highlighted with a red box), 'Date of birth' (mm/dd/yyyy), 'First name', 'Last name', 'Gender' (Female, Male, Other), and 'Reason for recording' (Not selected). There is a 'Search' button next to the CPR number field. Below these fields is a 'Notes on patient or recording' text area. At the bottom, there is a 'Device setup' section with 'Recording length' (1 day), 'Pacemaker' (None), and 'Sample rate' (256 Hz). A warning message at the bottom states: 'This will erase all recordings on the device'. There are 'Cancel' and 'Save to device' buttons.

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Patient

CPR number

Date of birth

mm/dd/yyyy

Search

First name

Last name

Gender

Female Male Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None

Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

⚠ This will erase all recordings on the device

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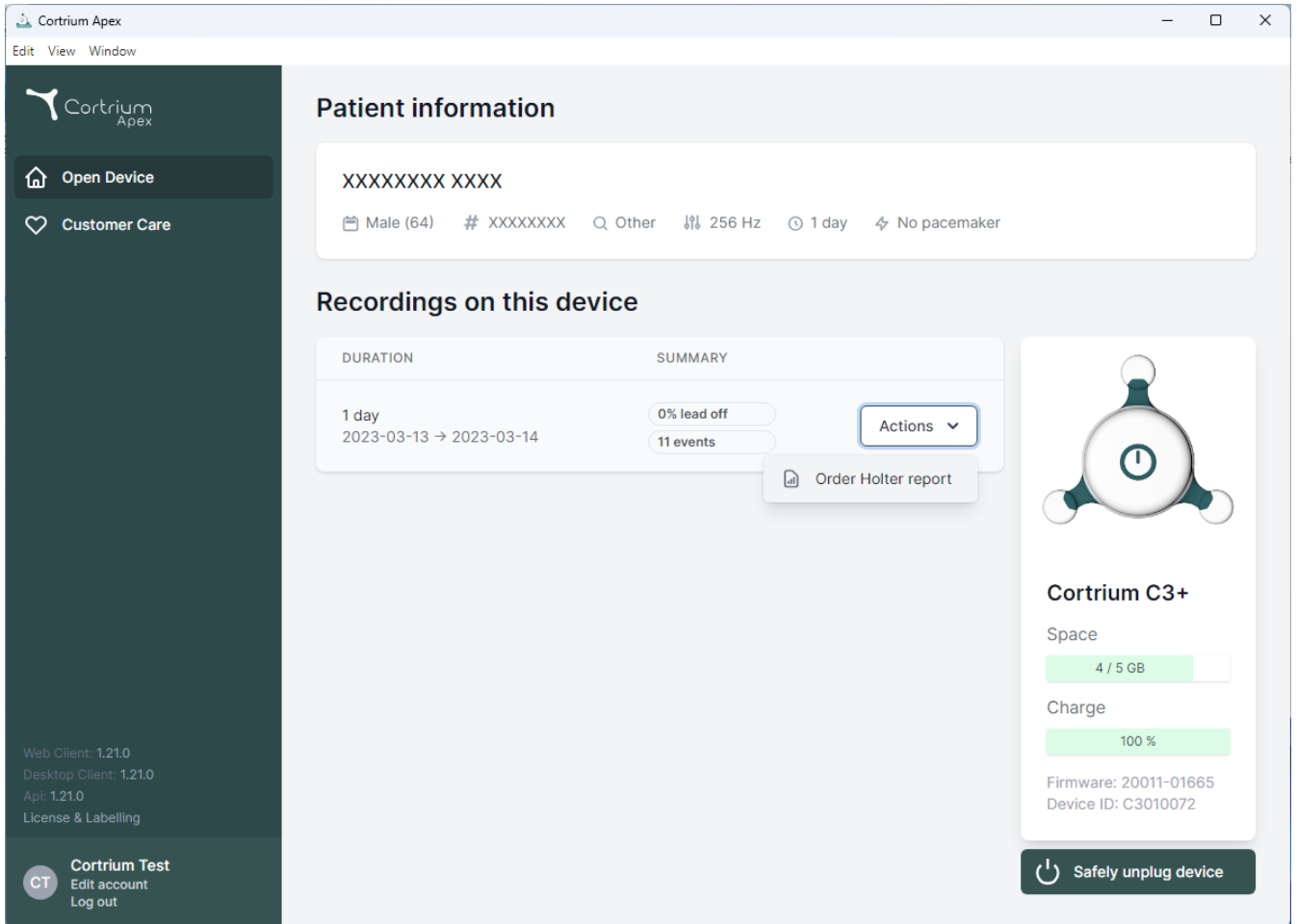
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device', 'Customer Care', and version information (Web Client: 1.21.0, Desktop Client: 1.21.0, Api: 1.21.0, License & Labelling). The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

**Patient information**

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  |
|----------------------------------|--------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events |

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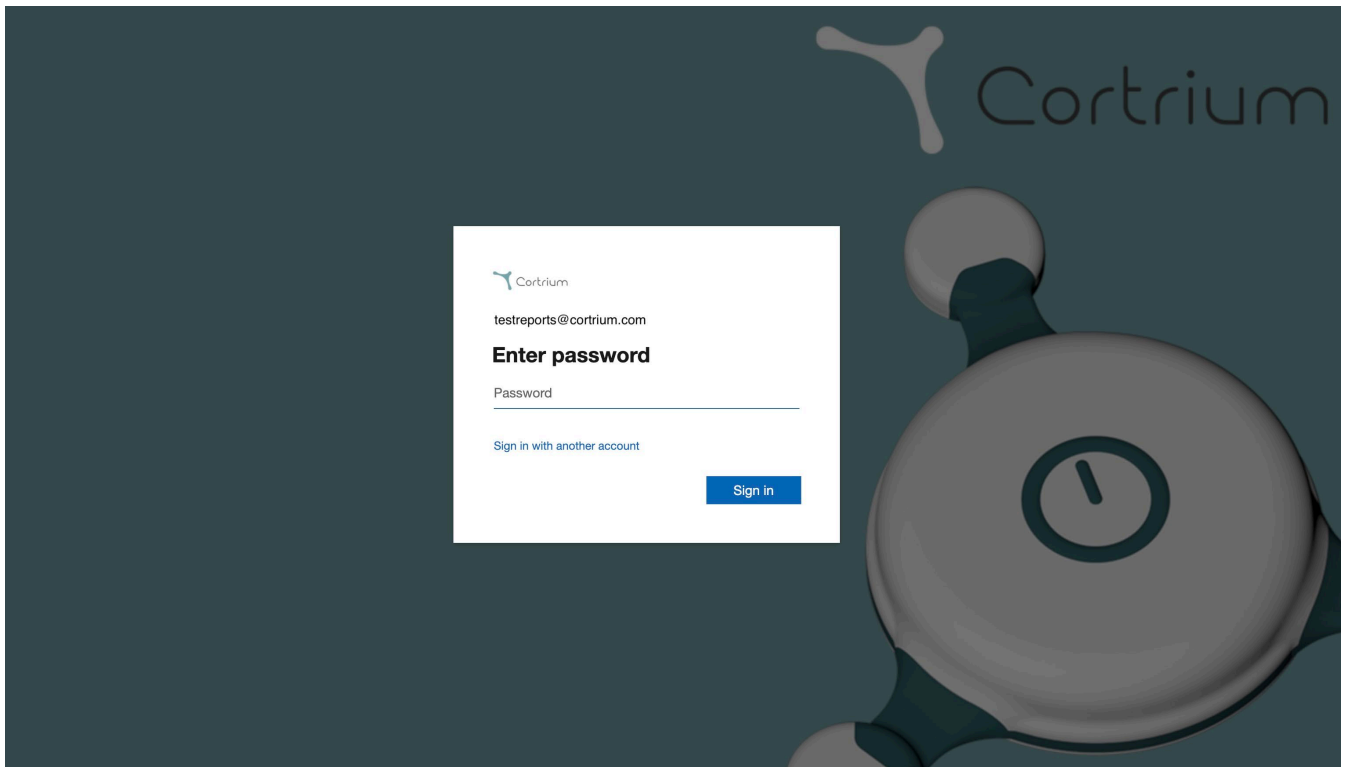
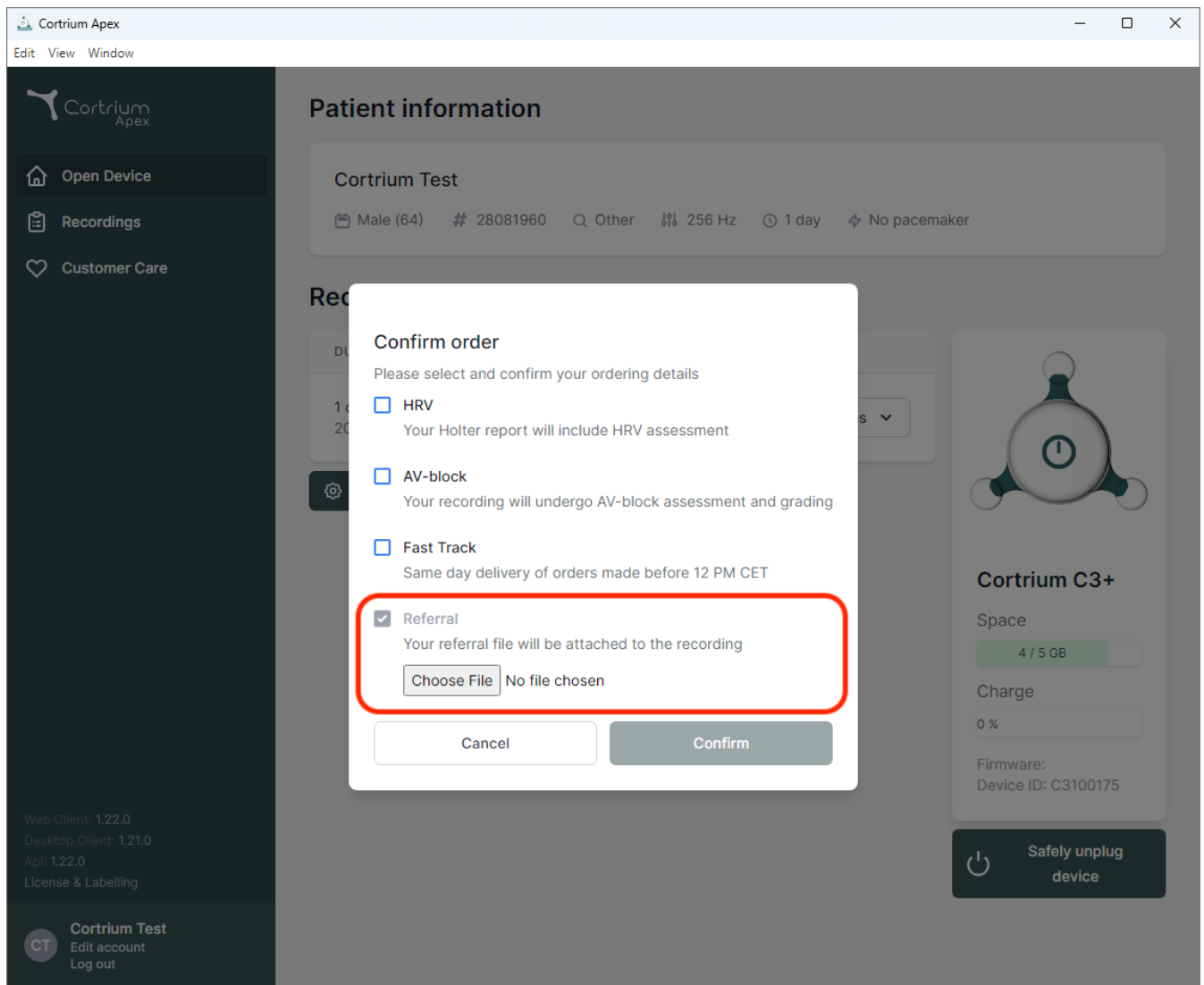
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
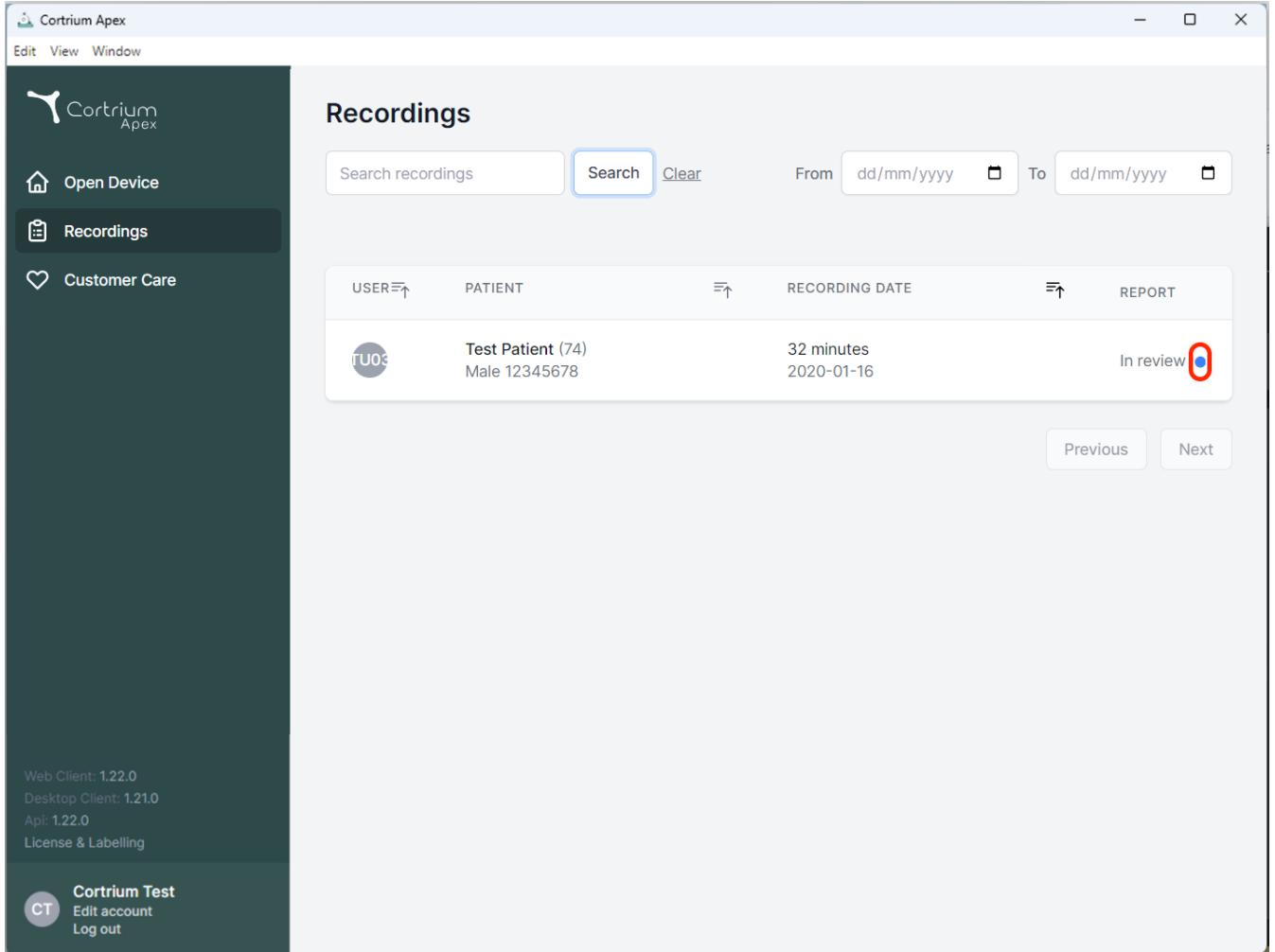
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (selected), and "Customer Care". The main content area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a status of "In review". A blue dot next to "In review" is circled in red. At the bottom right of the table are "Previous" and "Next" buttons. The bottom of the sidebar shows version information and a user profile for "Cortrium Test".

| USER | PATIENT                            | RECORDING DATE           | REPORT    |
|------|------------------------------------|--------------------------|-----------|
| TU03 | Test Patient (74)<br>Male 12345678 | 32 minutes<br>2020-01-16 | In review |

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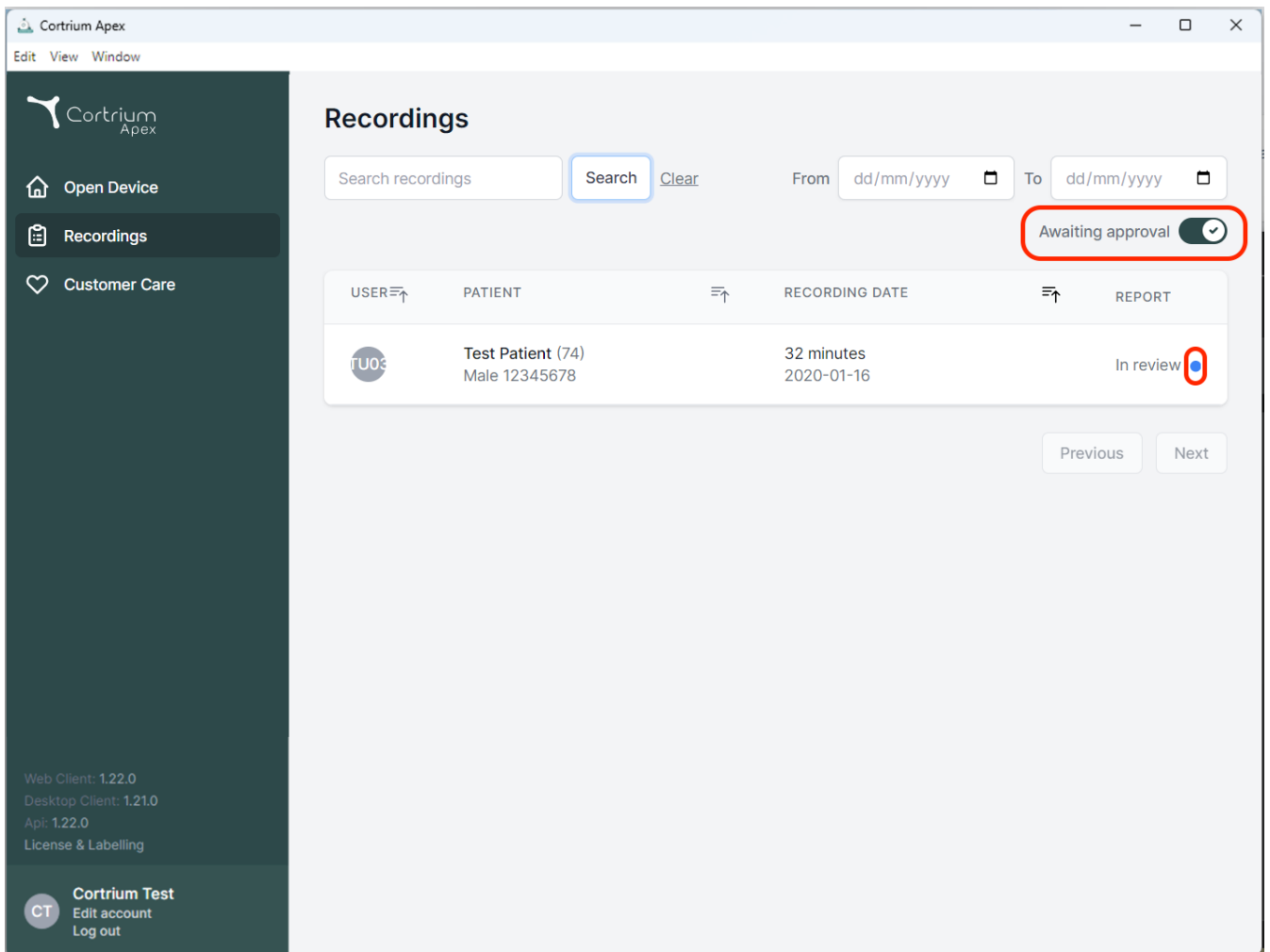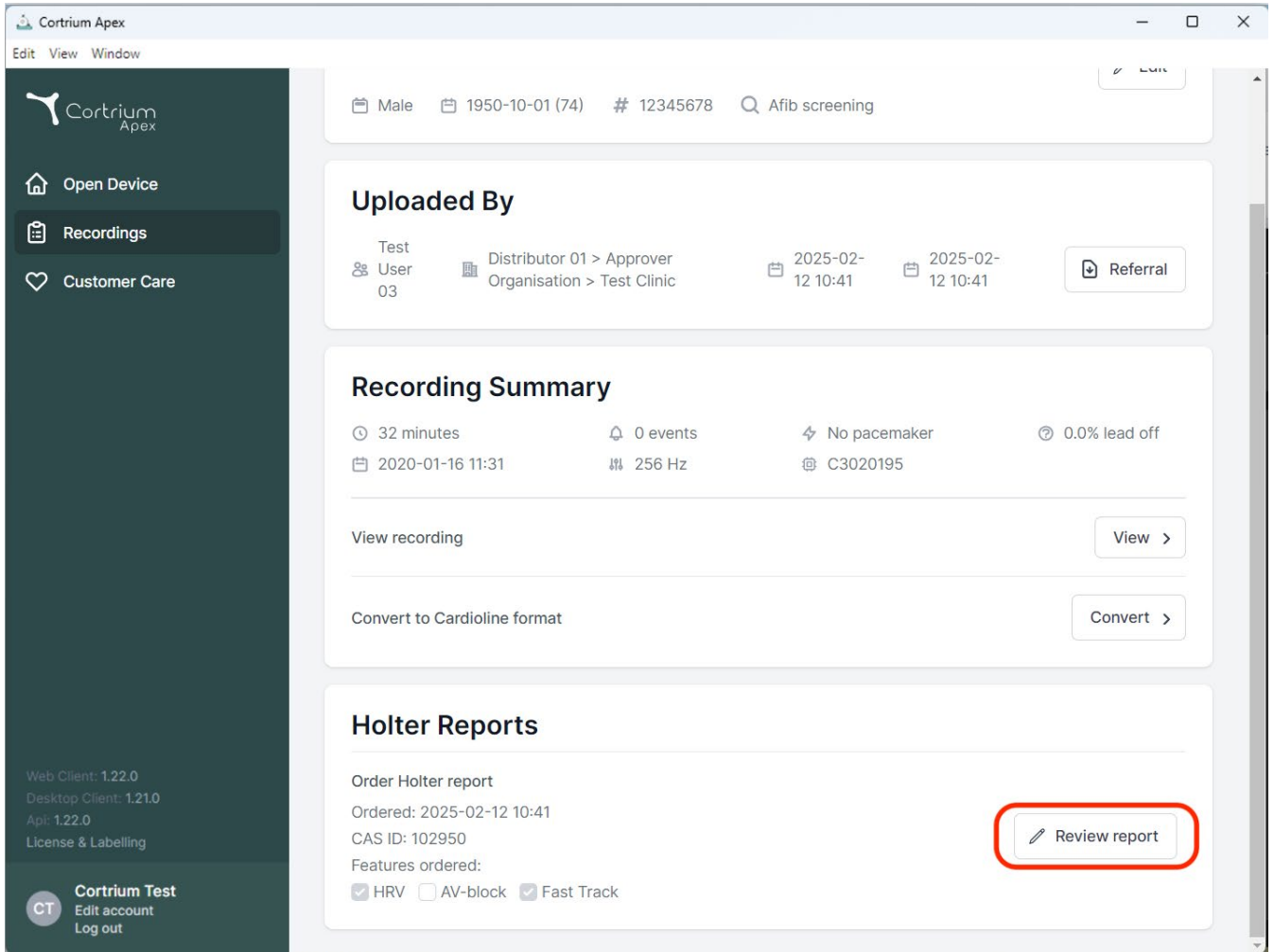
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and three main menu items: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. Below these, version information is listed: 'Web Client: 1.22.0', 'Desktop Client: 1.21.0', 'Api: 1.22.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes 'Edit account' and 'Log out' links.

The main content area shows patient information at the top: 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'. Below this, the 'Uploaded By' section displays 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic', with upload dates of '2025-02-12 10:41' and a 'Referral' button.

The 'Recording Summary' section provides details about the recording: '32 minutes', '0 events', 'No pacemaker', and '0.0% lead off'. It also shows the recording date '2020-01-16 11:31', a frequency of '256 Hz', and a device ID 'C3020195'. Action buttons include 'View recording', 'View >', 'Convert to Cardioline format', and 'Convert >'.

The 'Holter Reports' section shows the 'Order Holter report' details: 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A red box highlights the 'Review report' button.

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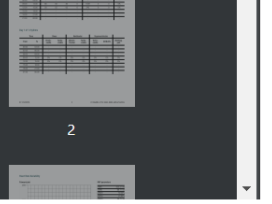
Summary

Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis. The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1). The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.  
Name of cardiologist: Test User 02.  
Notes: Some notes.

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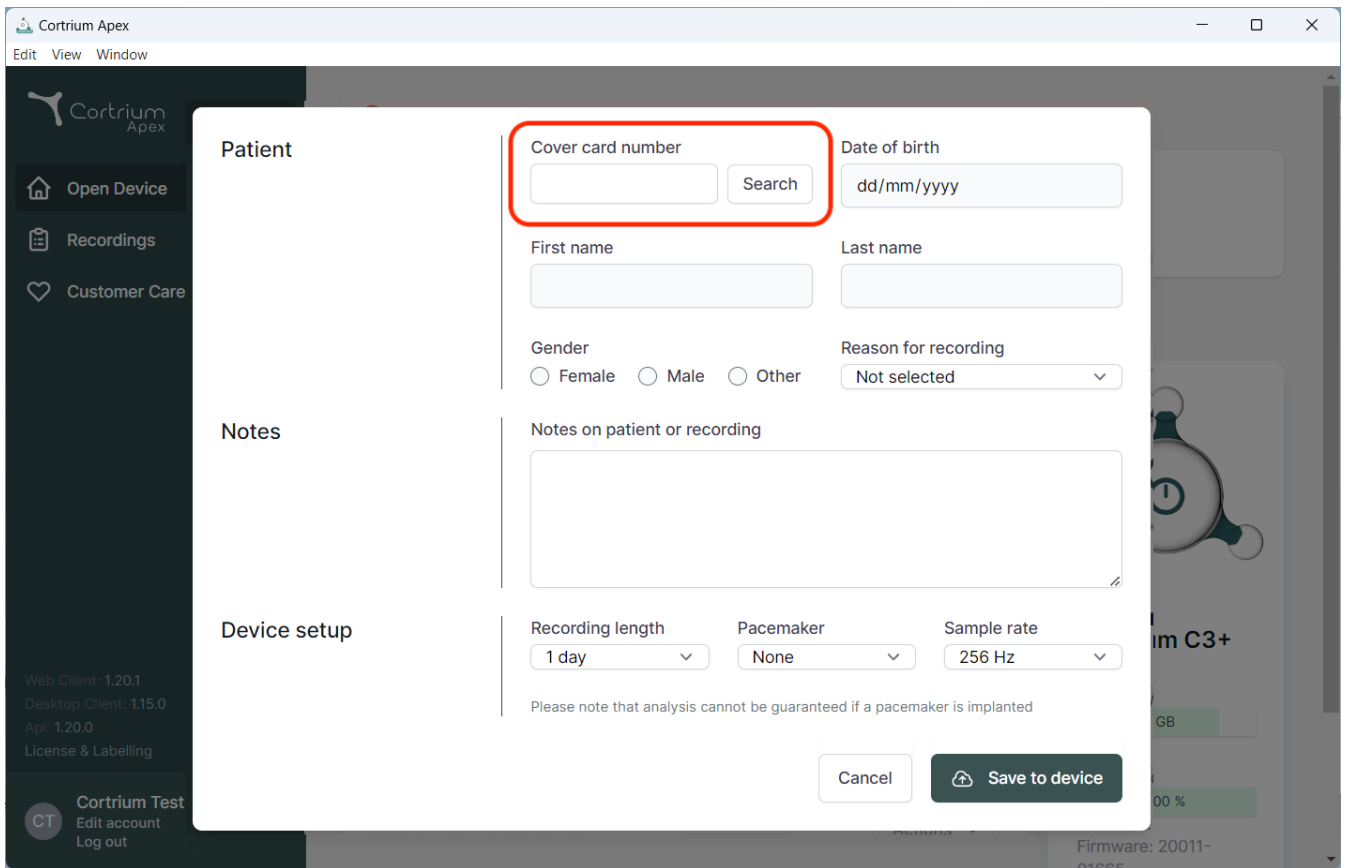
The screenshot shows the Cortrium Apex software interface. A dialog box titled "Patient" is open, allowing for patient search and device setup. The dialog box is divided into three main sections: "Patient", "Notes", and "Device setup".

**Patient Section:**

- Cover card number:** A text input field with a "Search" button next to it. This section is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

**Notes Section:**

- Notes on patient or recording:** A large text area for entering notes.

**Device setup Section:**

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup options, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the dialog box, there are two buttons: "Cancel" and "Save to device".

The background of the software interface shows a sidebar with options: "Open Device", "Recordings", and "Customer Care". At the bottom left, there is a "Cortrium Test" section with links for "Edit account" and "Log out".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Tanska  
Sähköposti: [info@cortrium.com](mailto:info@cortrium.com) • Verkkosivusto: [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033

CE  
0123  
46 / 46