Cortrium Apex 1.27

Cortrium Apex 1.27

Mode d'emploi

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
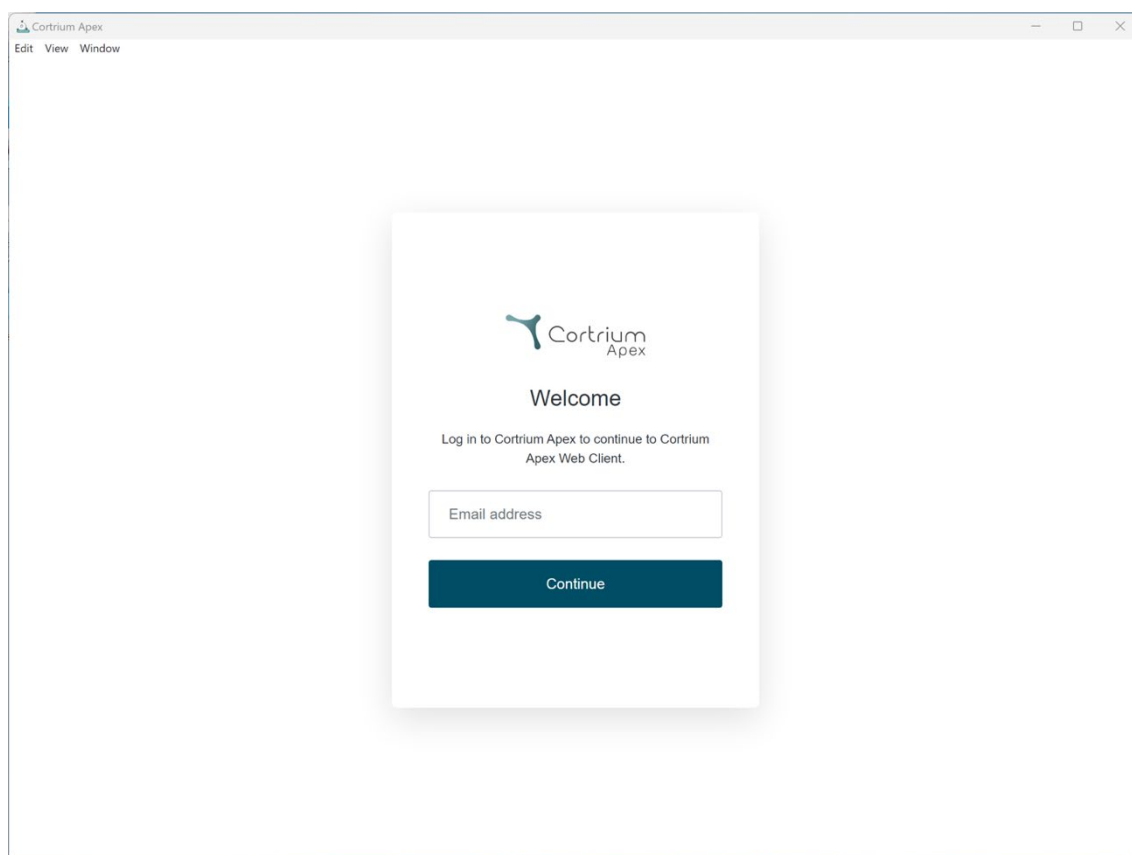
The screenshot shows a web browser window titled "Cortrium Apex". The browser's menu bar includes "Edit", "View", and "Window". In the center of the browser is a white login card with a subtle shadow. The card features the Cortrium Apex logo at the top, followed by the heading "Welcome". Below the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." There is a text input field labeled "Email address" and a dark blue button labeled "Continue" below it.

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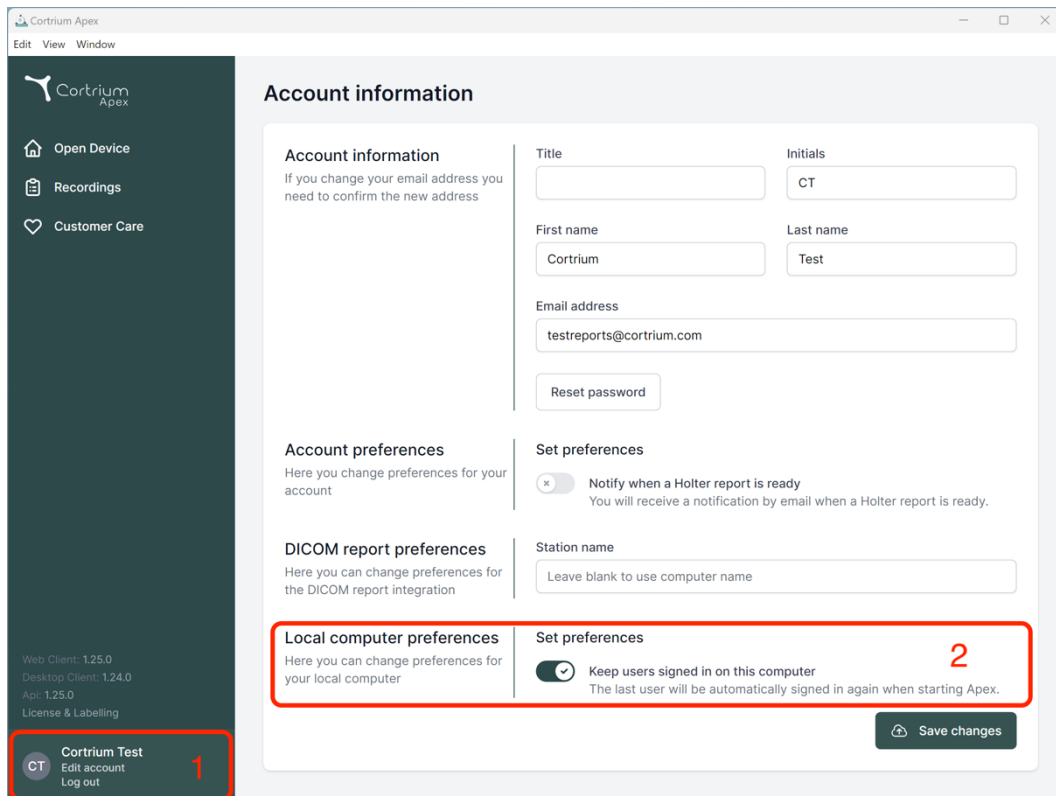The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details: 'Web Client: 1.25.0', 'Desktop Client: 1.24.0', 'Api: 1.25.0', and 'License & Labelling'. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with options to 'Edit account' and 'Log out', marked with a red '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' for changing account settings; 'DICOM report preferences' for integration settings; and 'Local computer preferences' for local device settings. The 'Local computer preferences' section includes a toggle for 'Keep users signed in on this computer', which is currently turned on and marked with a red '2'. A 'Save changes' button is located at the bottom right of the preferences section.

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Station name

Local computer preferences

Here you can change preferences for your local computer

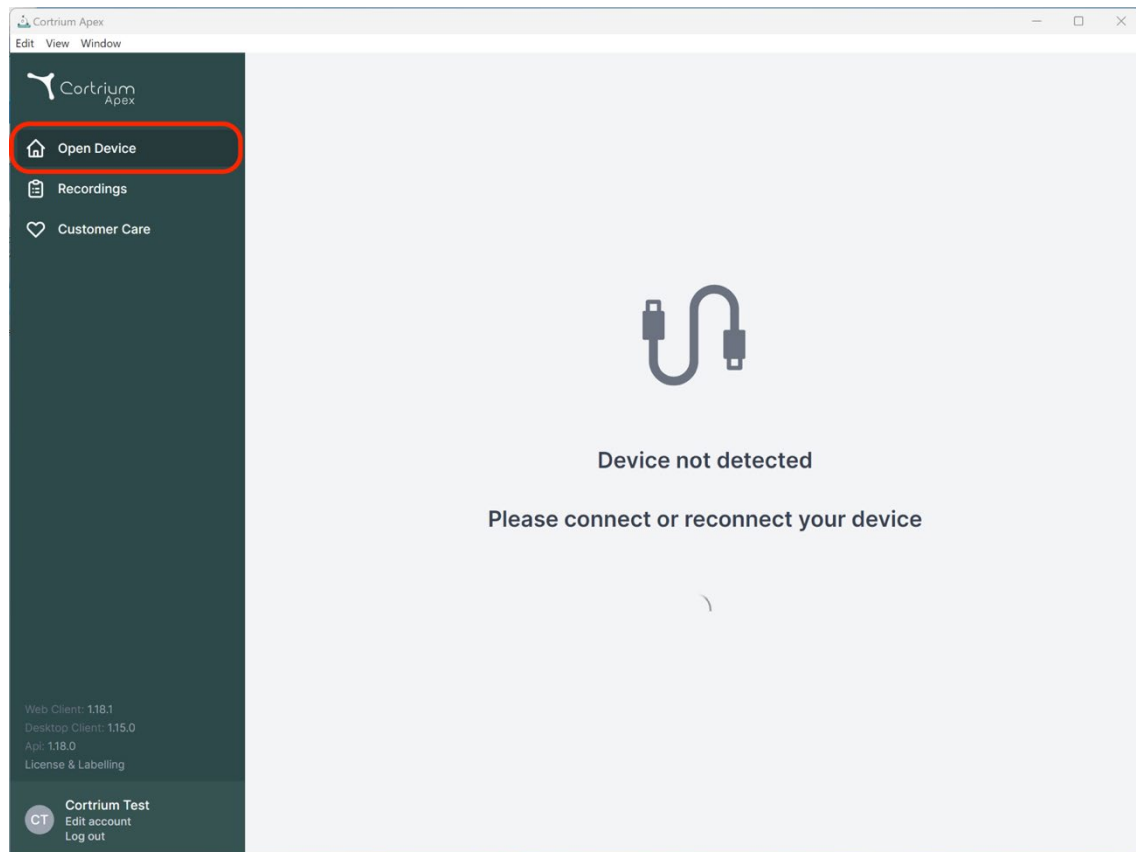
☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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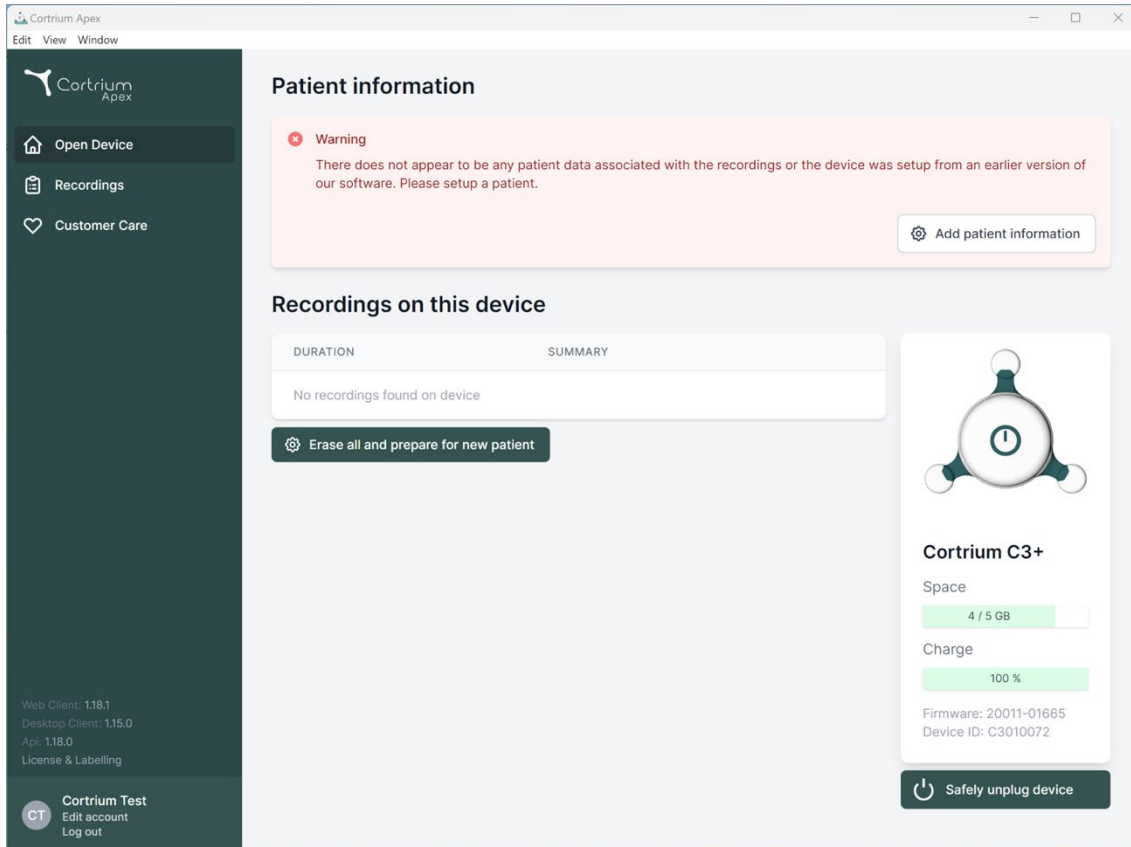
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area is titled 'Patient information' and features a red warning box stating: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is the 'Recordings on this device' section, which contains a table with columns 'DURATION' and 'SUMMARY'. The table is empty, showing 'No recordings found on device', and includes an 'Erase all and prepare for new patient' button. On the right side of the main area is a device status card for 'Cortrium C3+'. It displays a battery icon, 'Space' usage at '4 / 5 GB', 'Charge' at '100 %', and technical details: 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Save to device

Warning This will erase all recordings on the device

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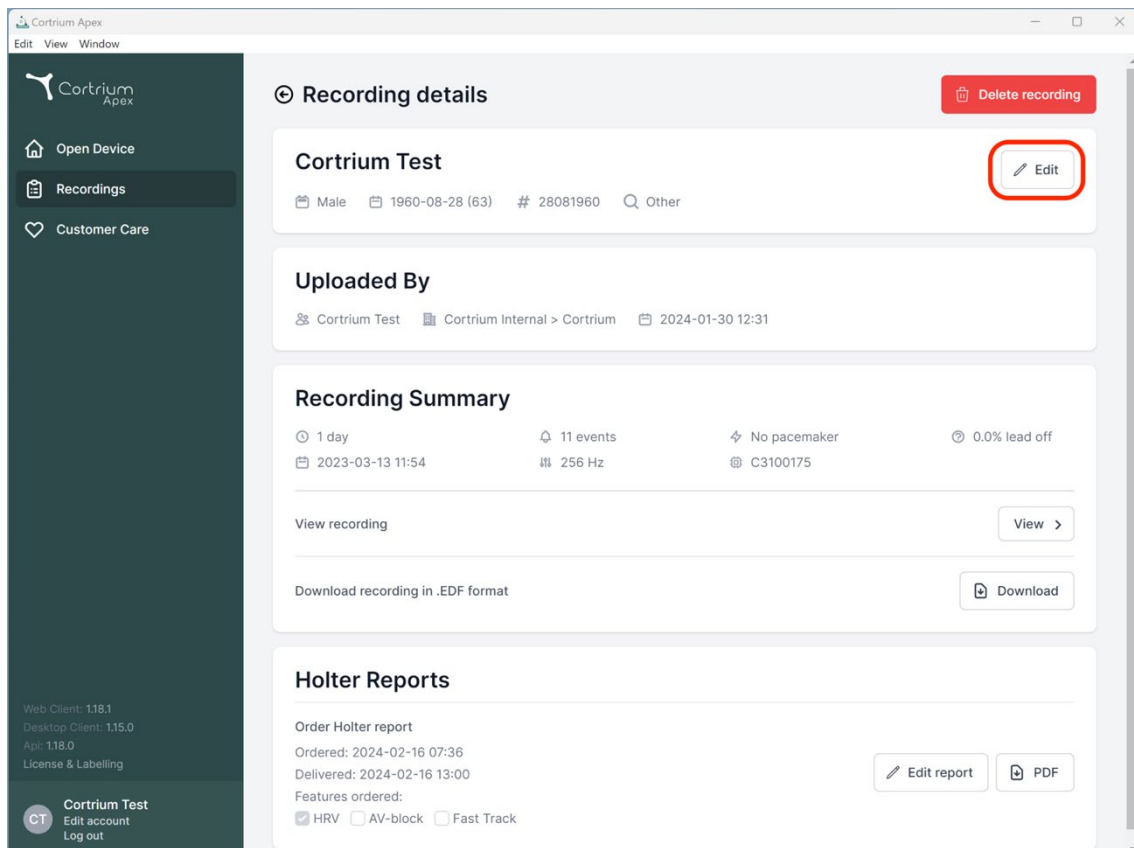
Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

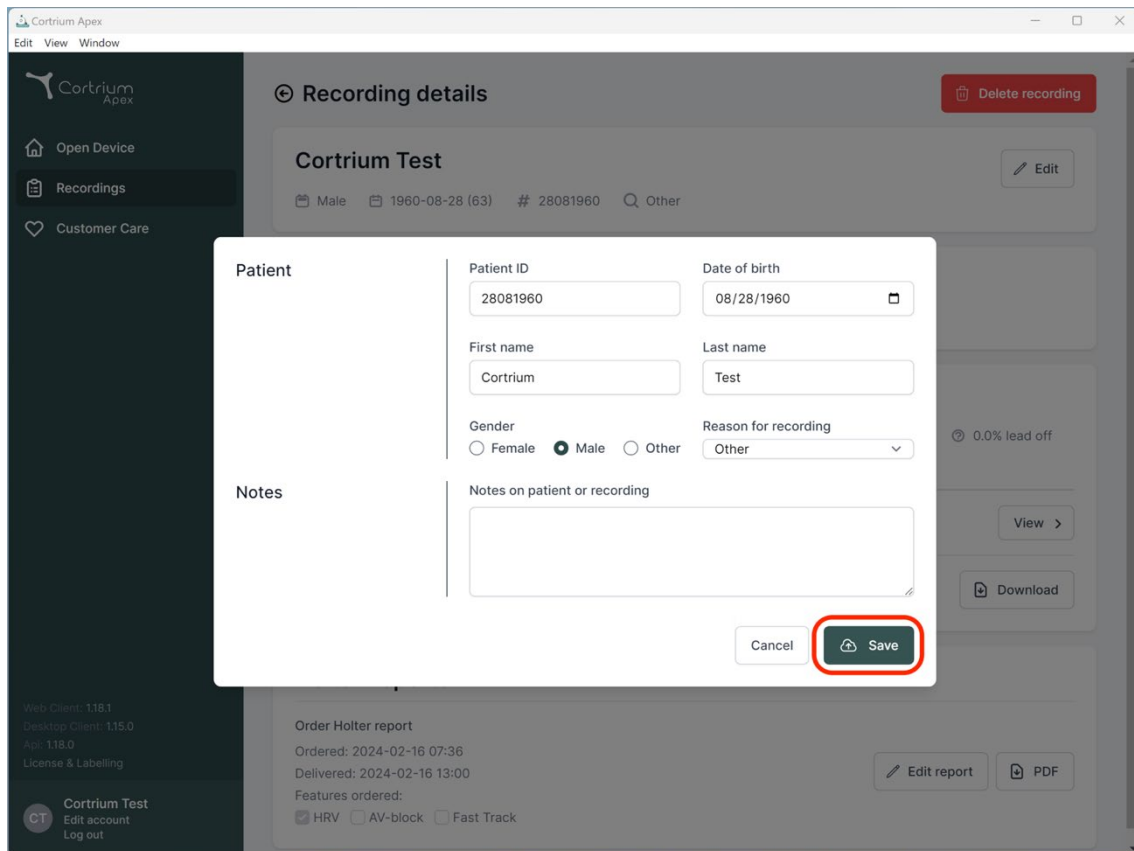
Recording Summary
 1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports
 Order Holter report
 Ordered: 2024-02-16 07:36
 Delivered: 2024-02-16 13:00
 Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Recording details" is open, showing information for a "Cortrium Test". The modal includes fields for Patient ID (28081960), Date of birth (08/28/1960), First name (Cortrium), Last name (Test), Gender (Male selected), Reason for recording (Other), and a Notes section. The "Save" button is highlighted with a red circle. The background shows the main interface with a sidebar containing "Open Device", "Recordings", and "Customer Care" options. The top right has a "Delete recording" button. The bottom section shows "Order Holter report" details, including "Ordered: 2024-02-16 07:36", "Delivered: 2024-02-16 13:00", and "Features ordered: HRV, AV-block, Fast Track".

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
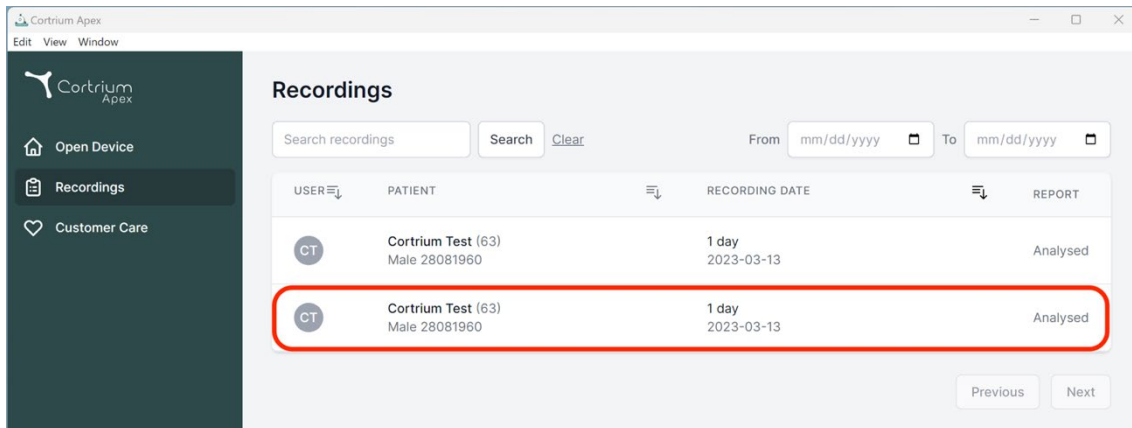
Recordings

Search recordings Search [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

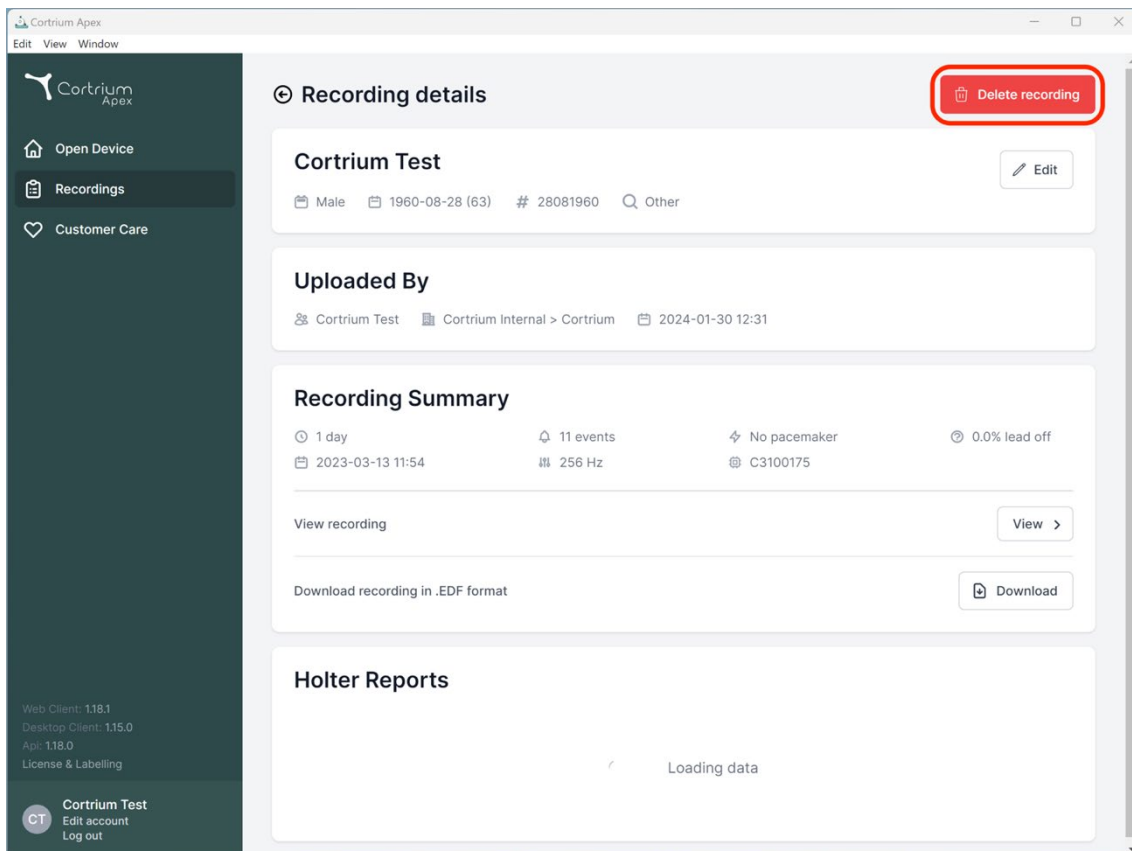Previous Next

Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

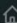
Loading data

Web Client: 1.18.1
Desktop Client: 1.15.0
Api: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

 Customer Care

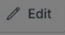
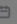
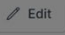
Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

 Cortrium Test
Edit account
Log out

Recording details

 Delete recording

Cortrium Test

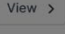
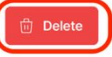
 Male  1960-08-28 (63)  # 28081960  Other  Edit

Uploaded By


 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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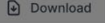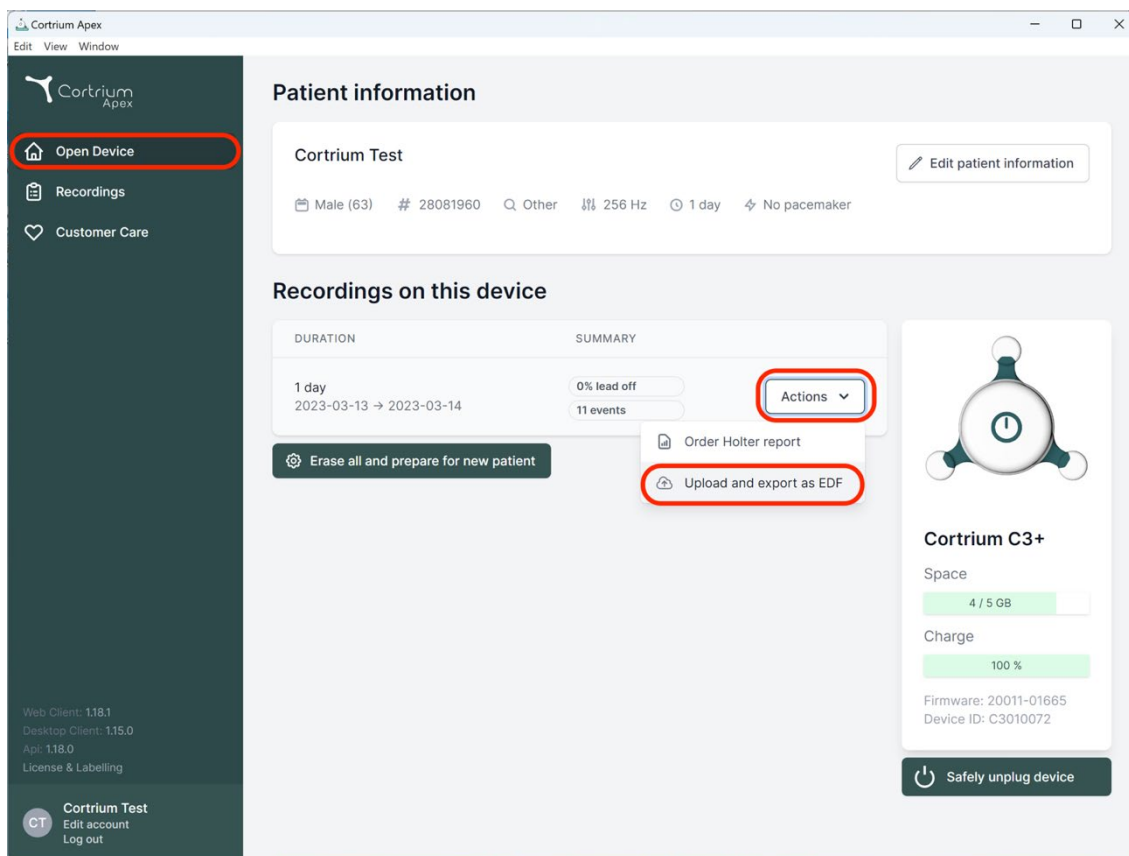
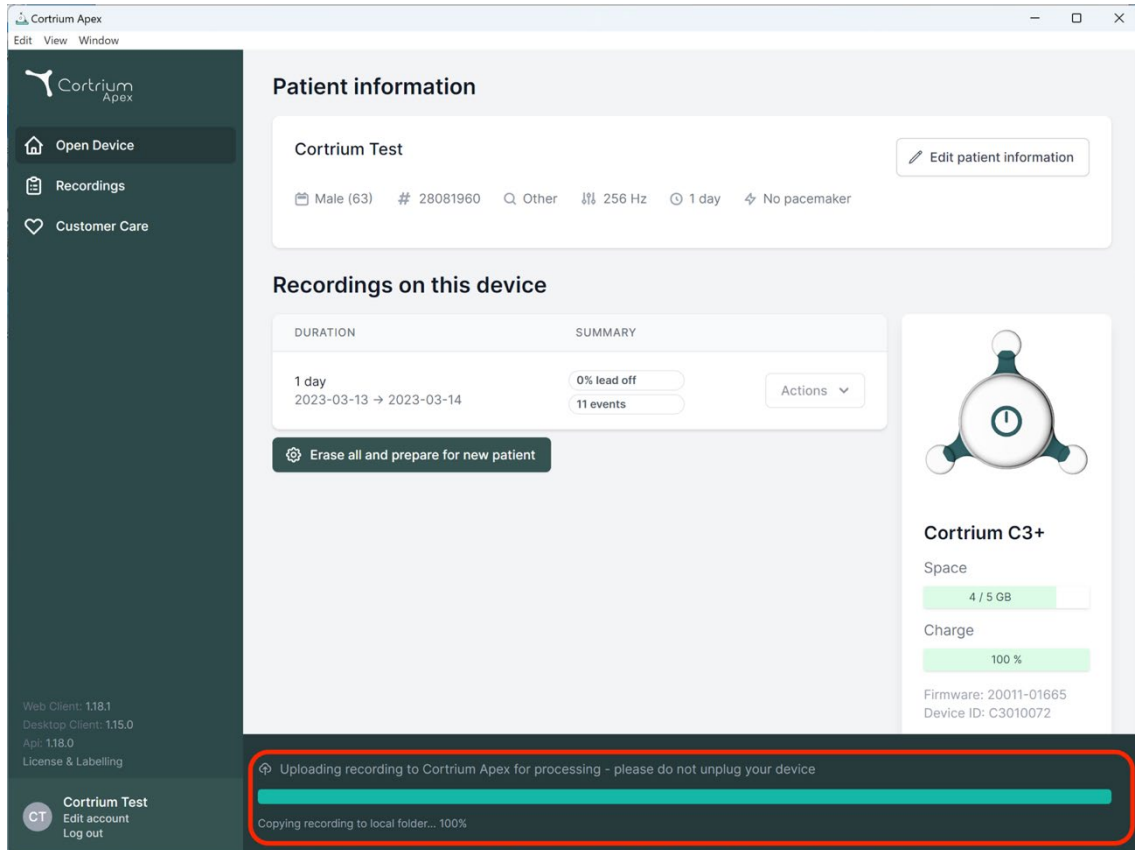
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

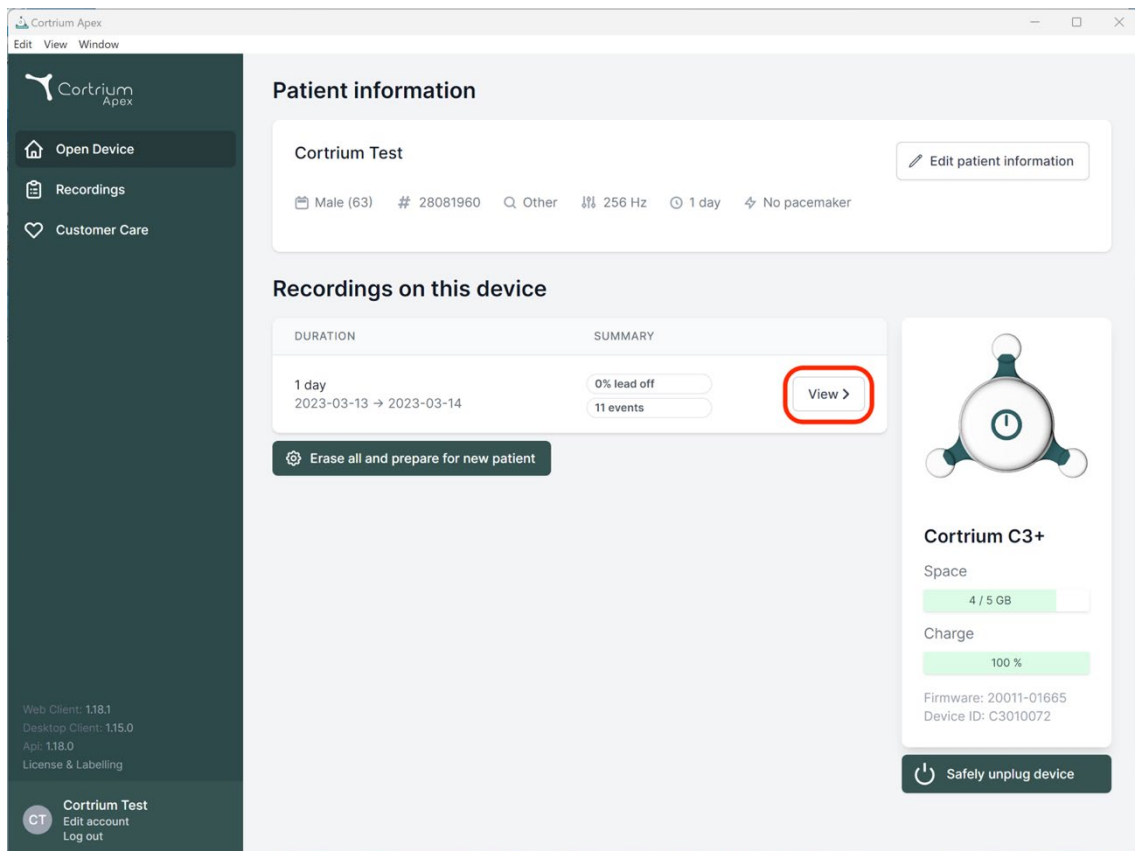
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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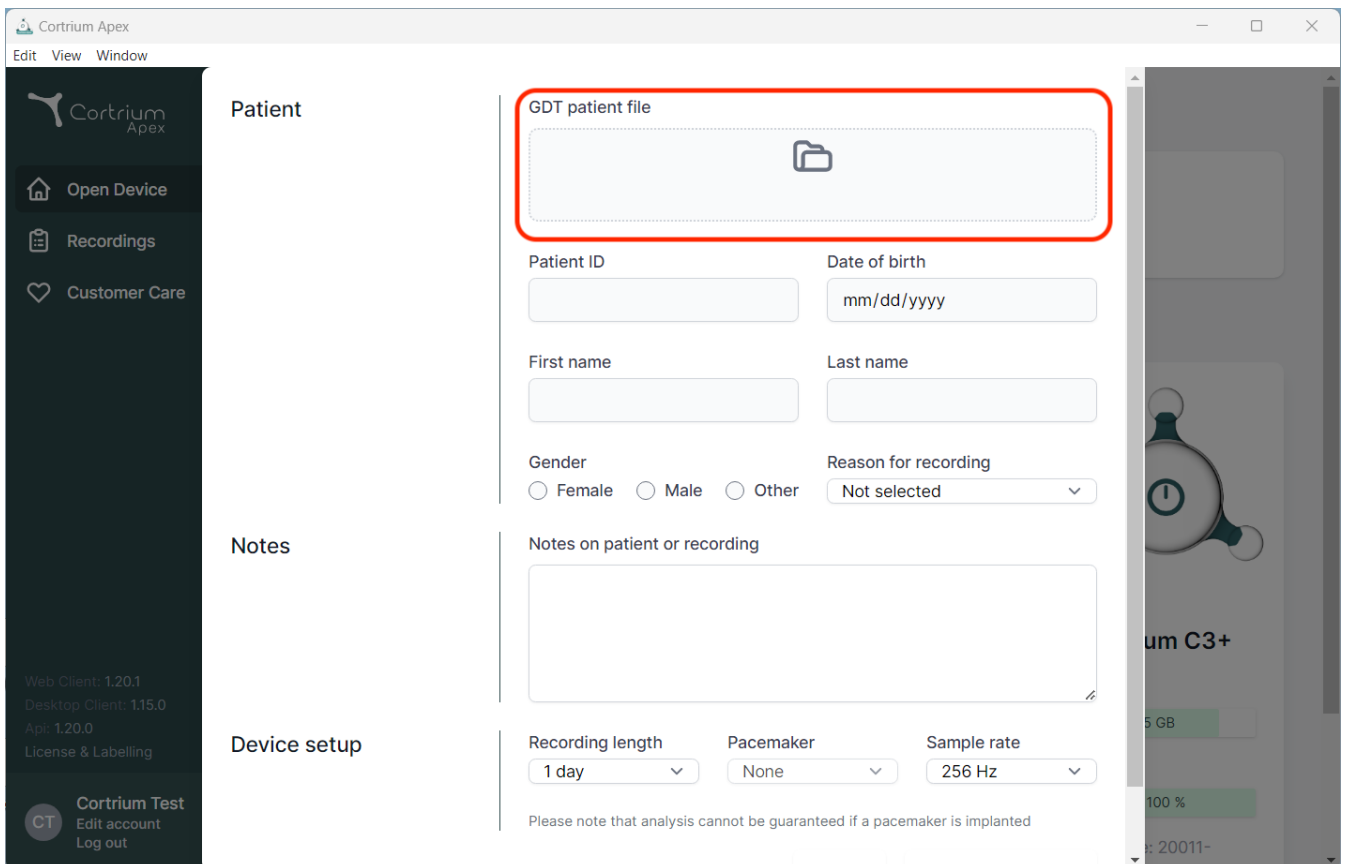
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

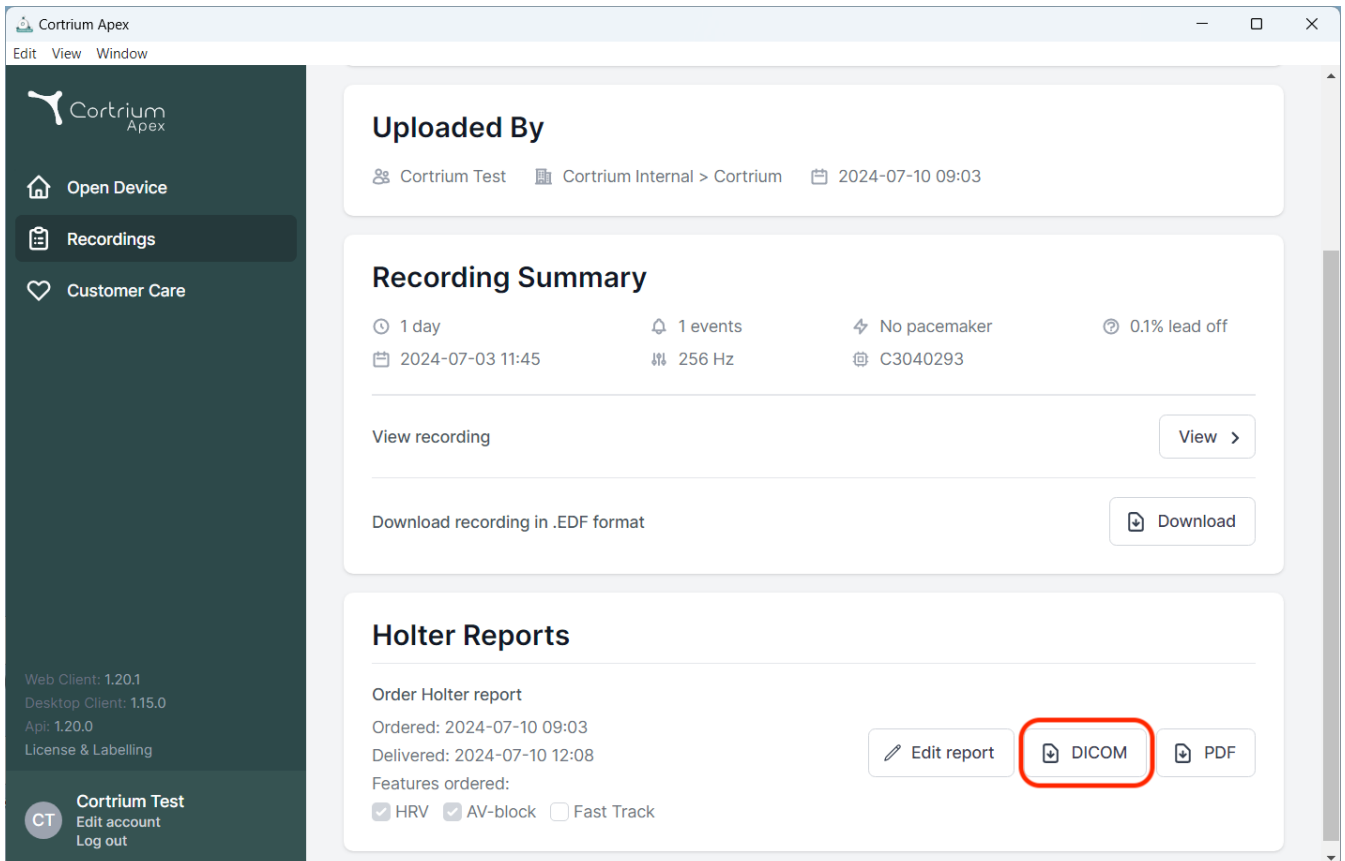
The main content area is divided into three sections:

- Patient:** This section contains a "GDT patient file" upload area, which is highlighted with a red rectangle. Below this are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected").
- Notes:** This section features a text area labeled "Notes on patient or recording".
- Device setup:** This section includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device screen is visible, showing a "Cortrium C3+" device with a battery level indicator at 100% and a date "20011-".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (for .EDF format).
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

The bottom of the sidebar contains version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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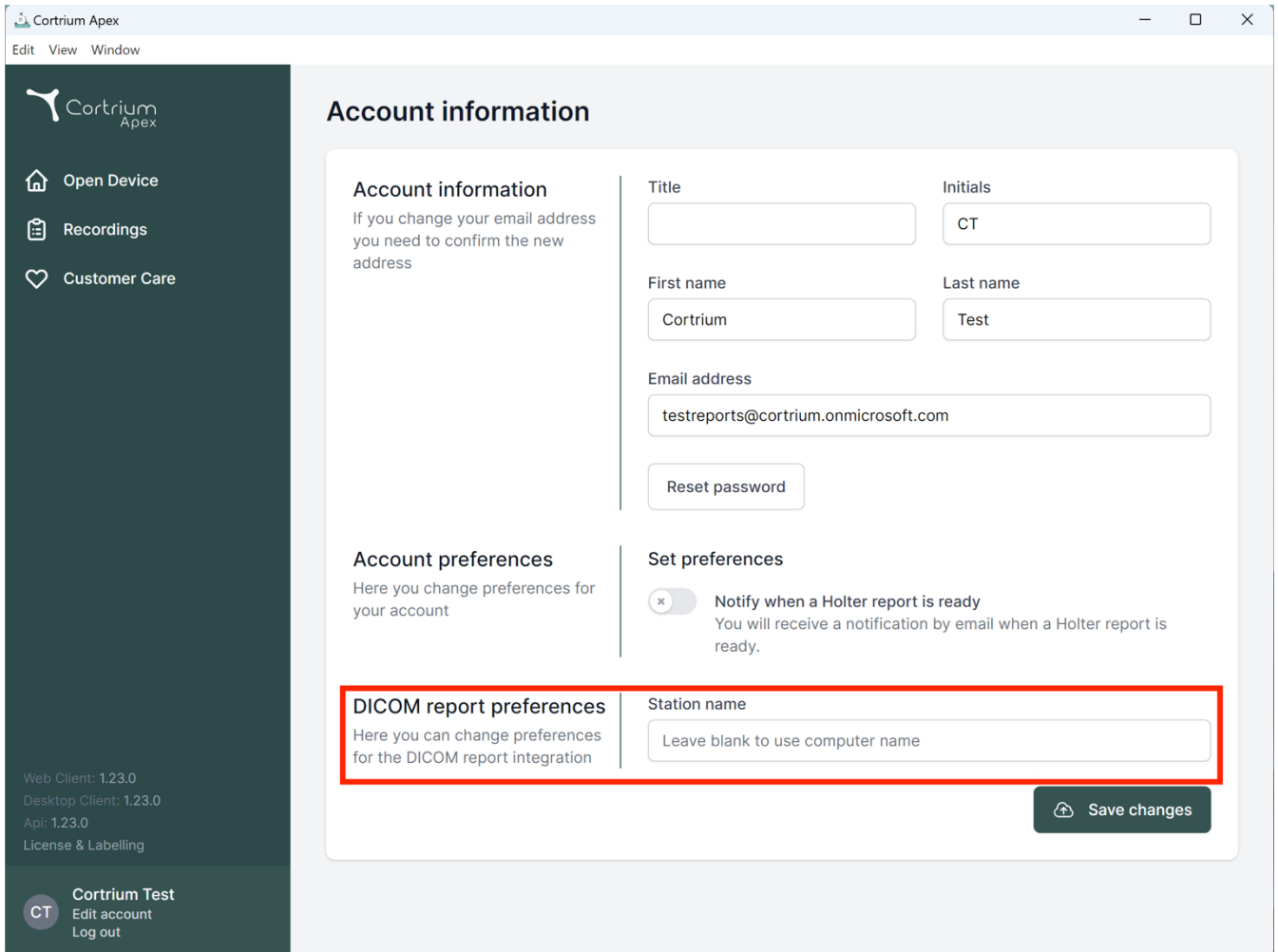Edit View Window

Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

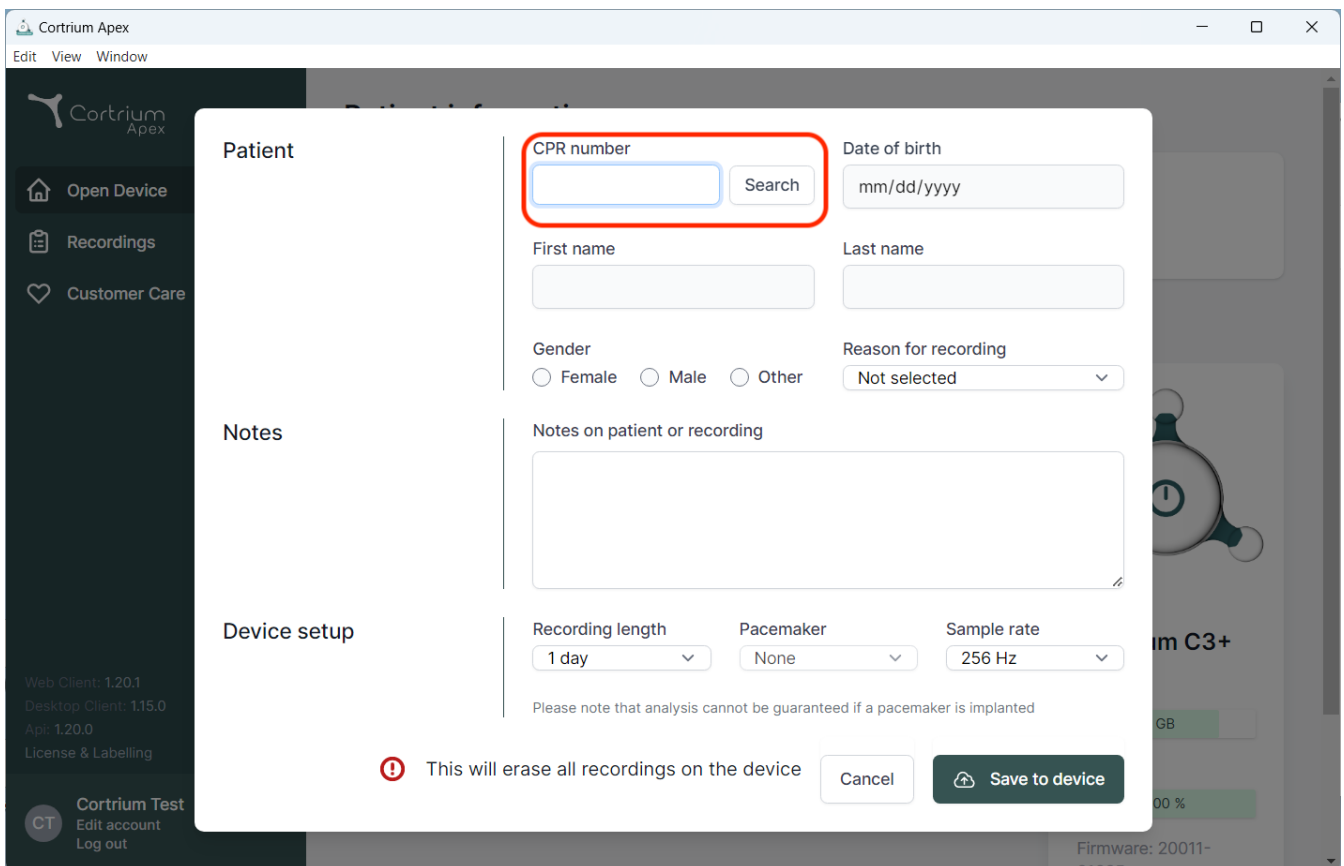
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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The screenshot shows the Cortrium Apex web application interface. A modal window is open for patient search. The modal is divided into three main sections: Patient, Notes, and Device setup.

Patient section:

- CPR number:** A text input field with a blue border, highlighted by a red rectangle. Next to it is a "Search" button.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Three radio buttons labeled "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current value.

Notes section:

- Notes on patient or recording:** A large text area for notes.

Device setup section:

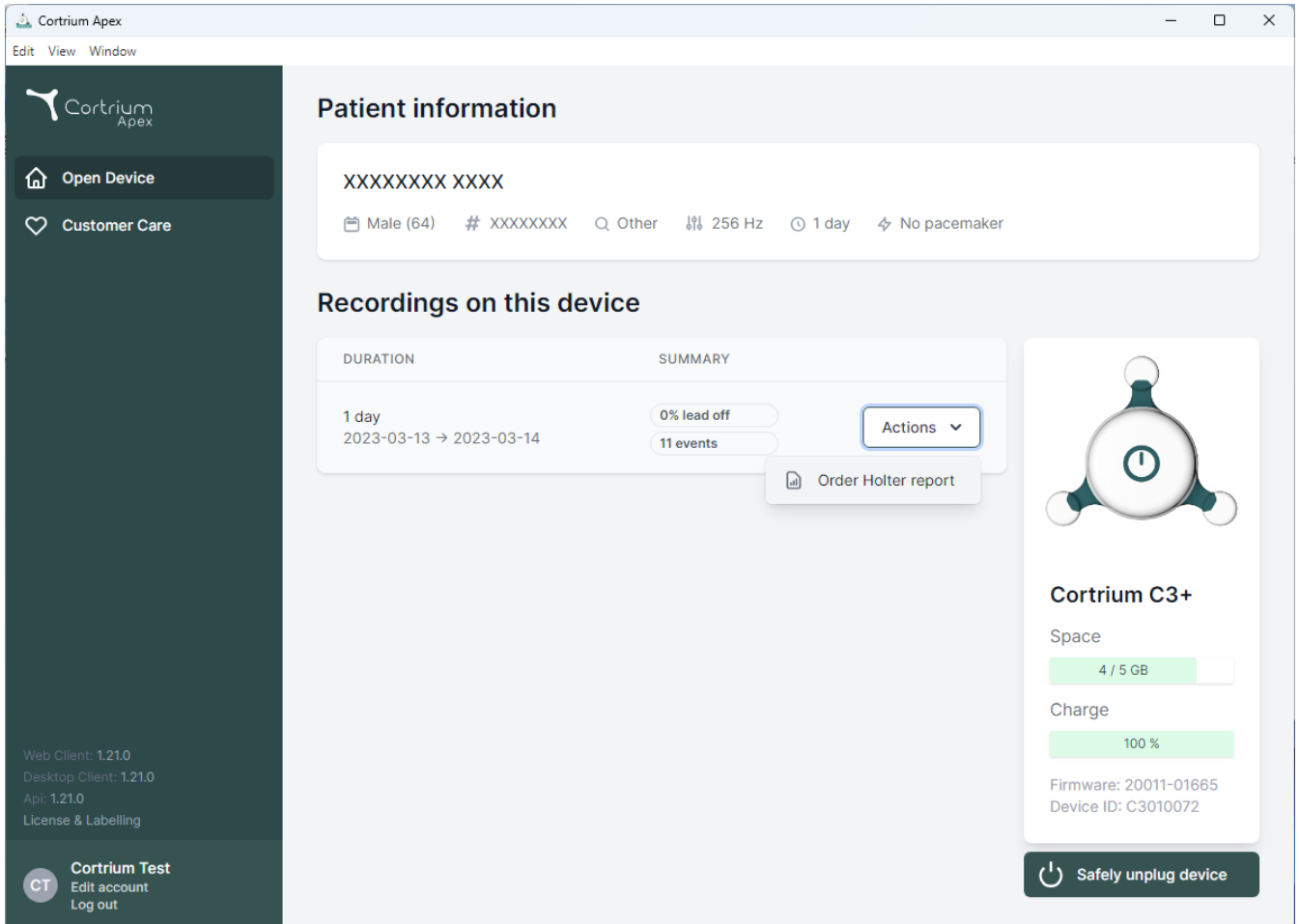
- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

Below the Device setup section, there is a warning message: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom of the modal, there is a red warning icon and the text: "This will erase all recordings on the device". To the right of this message are two buttons: "Cancel" and "Save to device".

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and two main menu items: 'Open Device' (with a house icon) and 'Customer Care' (with a heart icon). Below these, version information is listed: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar, a 'Cortrium Test' section includes a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section features a table with two columns: 'DURATION' and 'SUMMARY'. The table contains one entry for a '1 day' recording from '2023-03-13 → 2023-03-14', showing '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the recording, with an option to 'Order Holter report'. On the right side, a device status card for 'Cortrium C3+' shows 'Space' at '4 / 5 GB' and 'Charge' at '100 %'. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of the device card.

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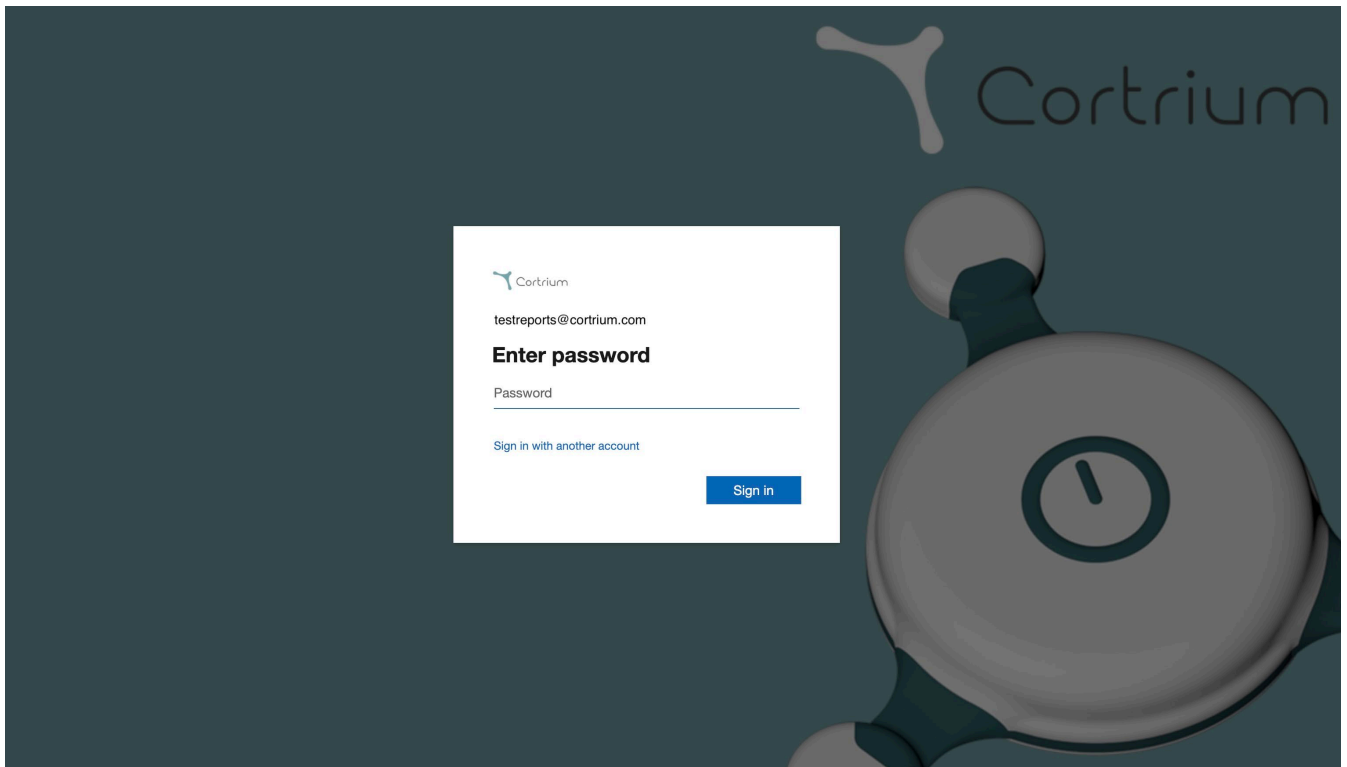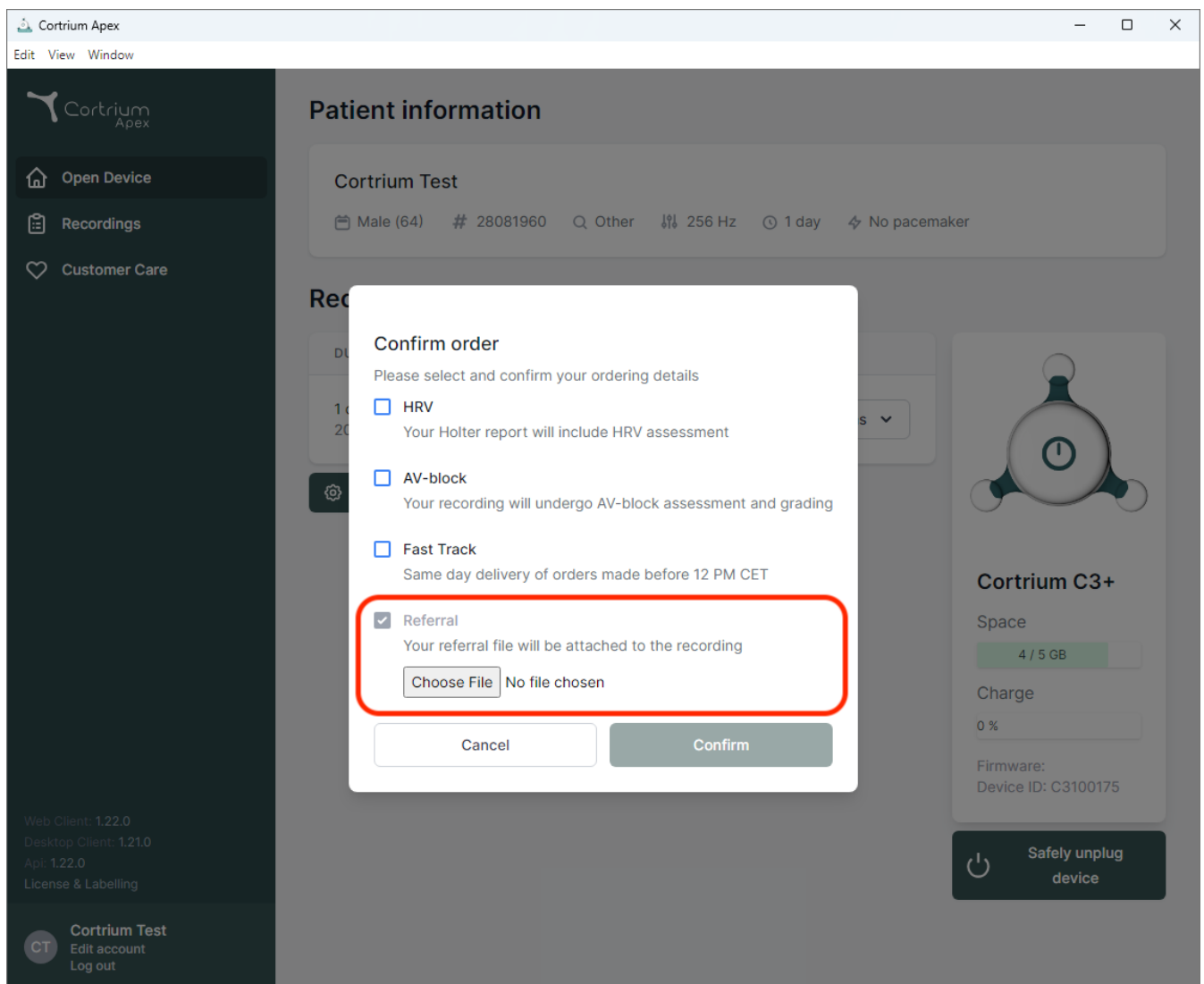
Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off

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
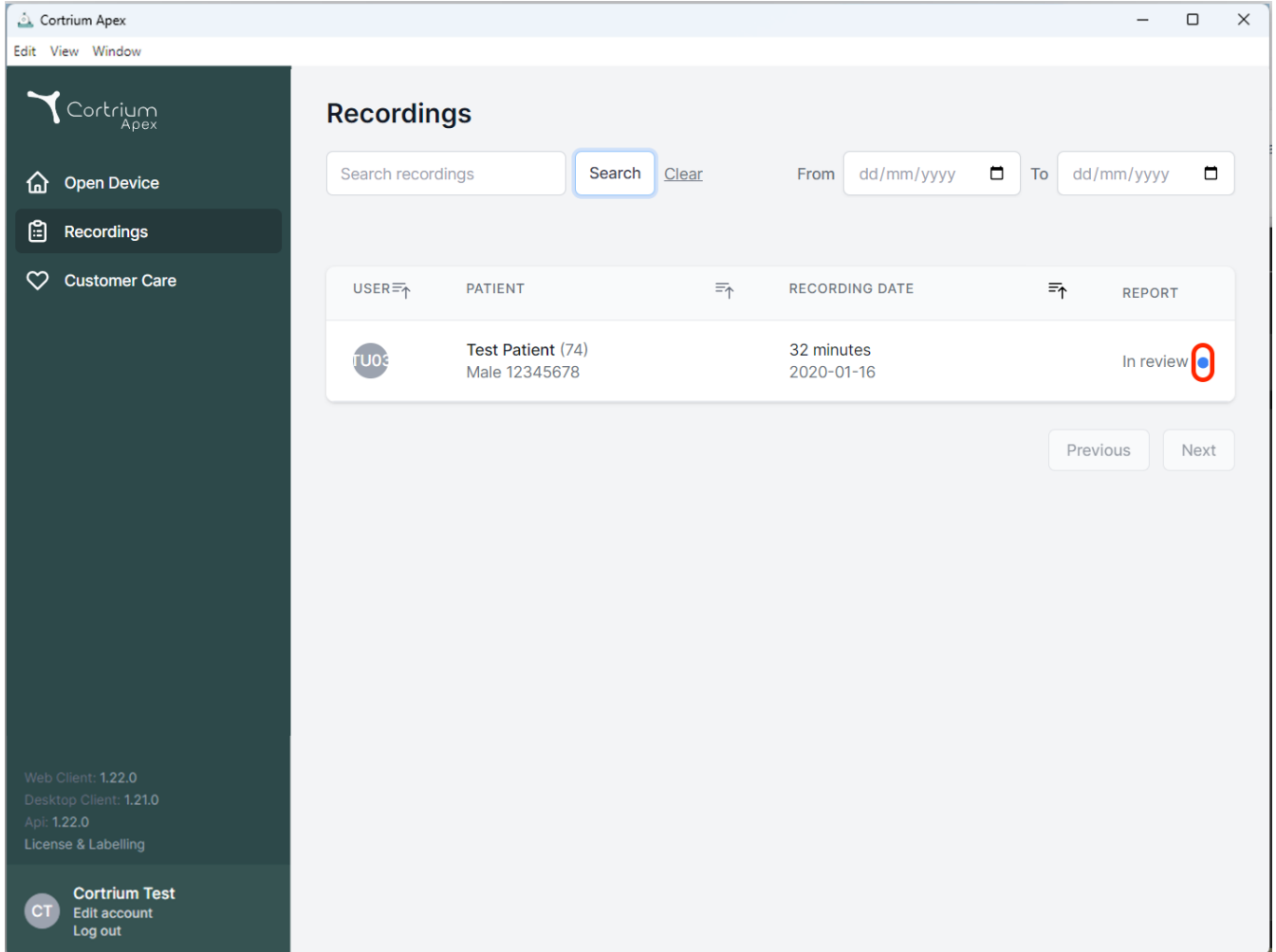
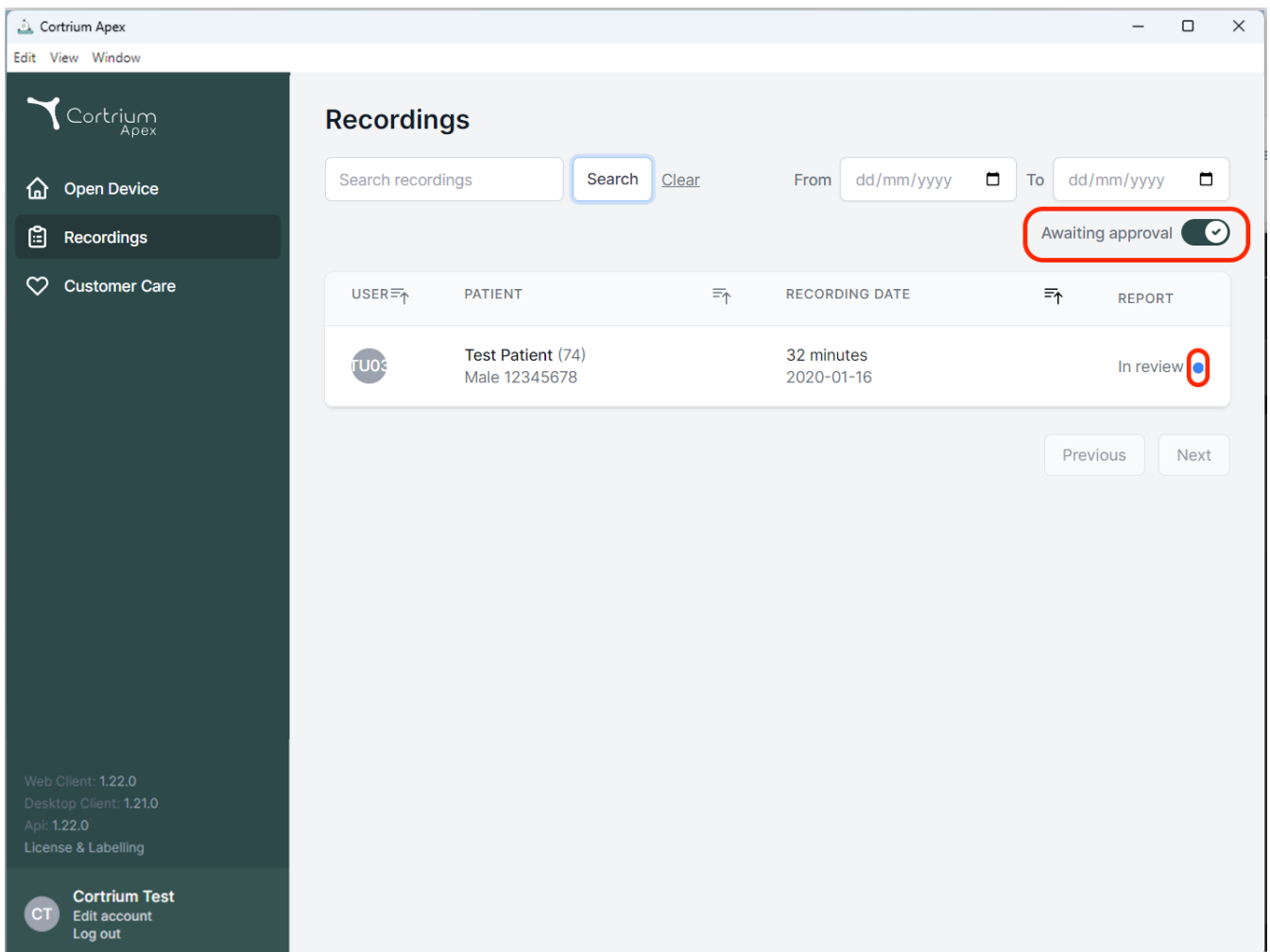
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main area is titled 'Recordings' and contains a search bar, date filters, and a table of recordings. A red box highlights the 'Awaiting approval' toggle switch, which is turned on. Another red box highlights a blue dot next to the 'In review' status in the table row for 'Test Patient (74)'.

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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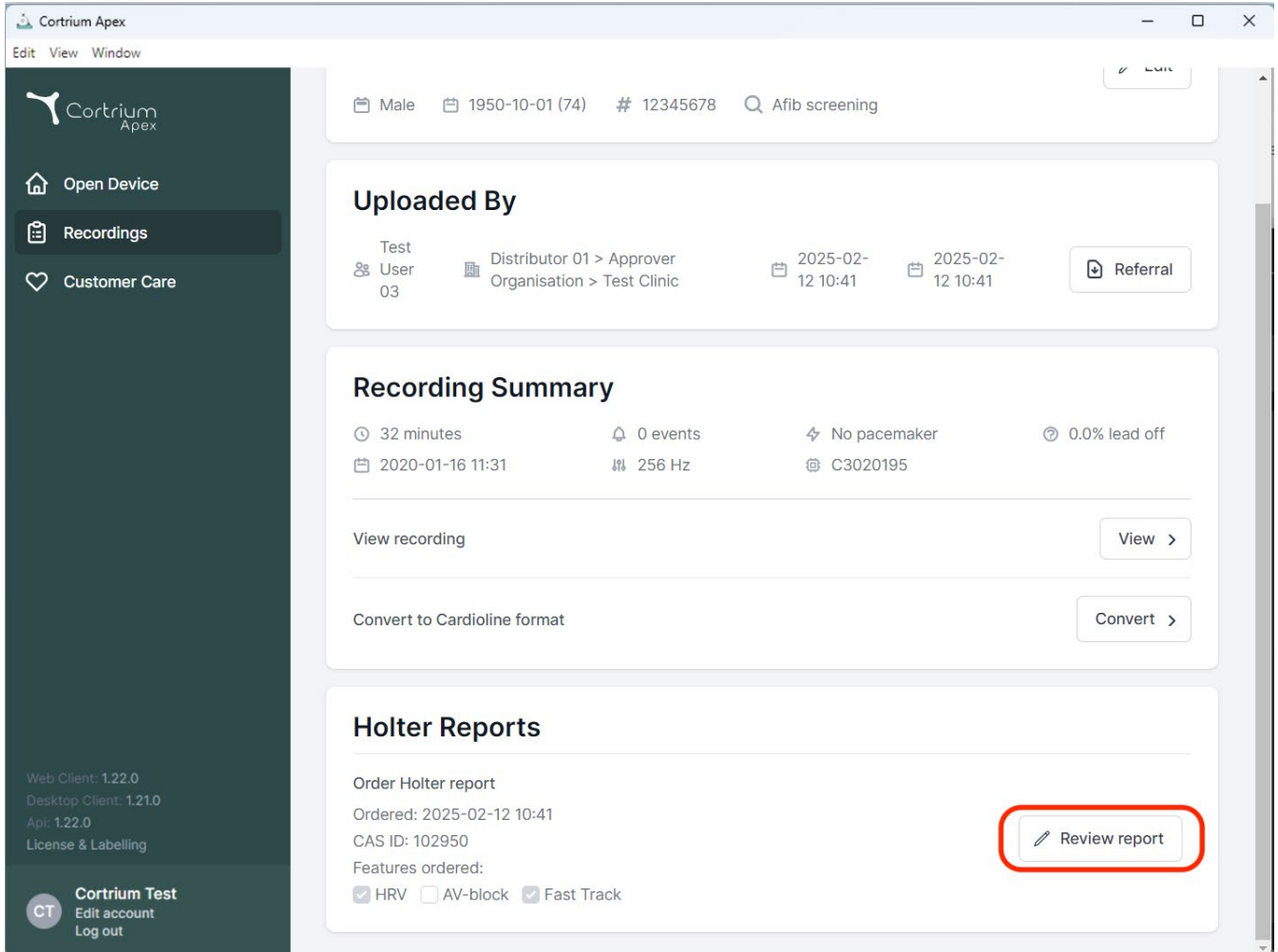
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: Open Device, Recordings (highlighted), and Customer Care. Below these are version details (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling) and a user profile for 'Cortrium Test' with options to edit account or log out.

The main content area shows patient information at the top: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the 'Uploaded By' section, indicating the test was performed by 'User 03' on '2025-02-12 10:41' for 'Distributor 01 > Approver Organisation > Test Clinic'. A 'Referral' button is present.

The 'Recording Summary' section provides details about the recording: 32 minutes duration, 0 events, No pacemaker, and 0.0% lead off. It also shows the recording date as '2020-01-16 11:31' and a frequency of '256 Hz'. There are buttons for 'View recording' and 'Convert to Cardioline format'.

The 'Holter Reports' section shows an 'Order Holter report' with an order date of '2025-02-12 10:41' and CAS ID '102950'. It lists ordered features: HRV (checked), AV-block (unchecked), and Fast Track (checked). A 'Review report' button, highlighted with a red rectangle, is located in the top right of this section.

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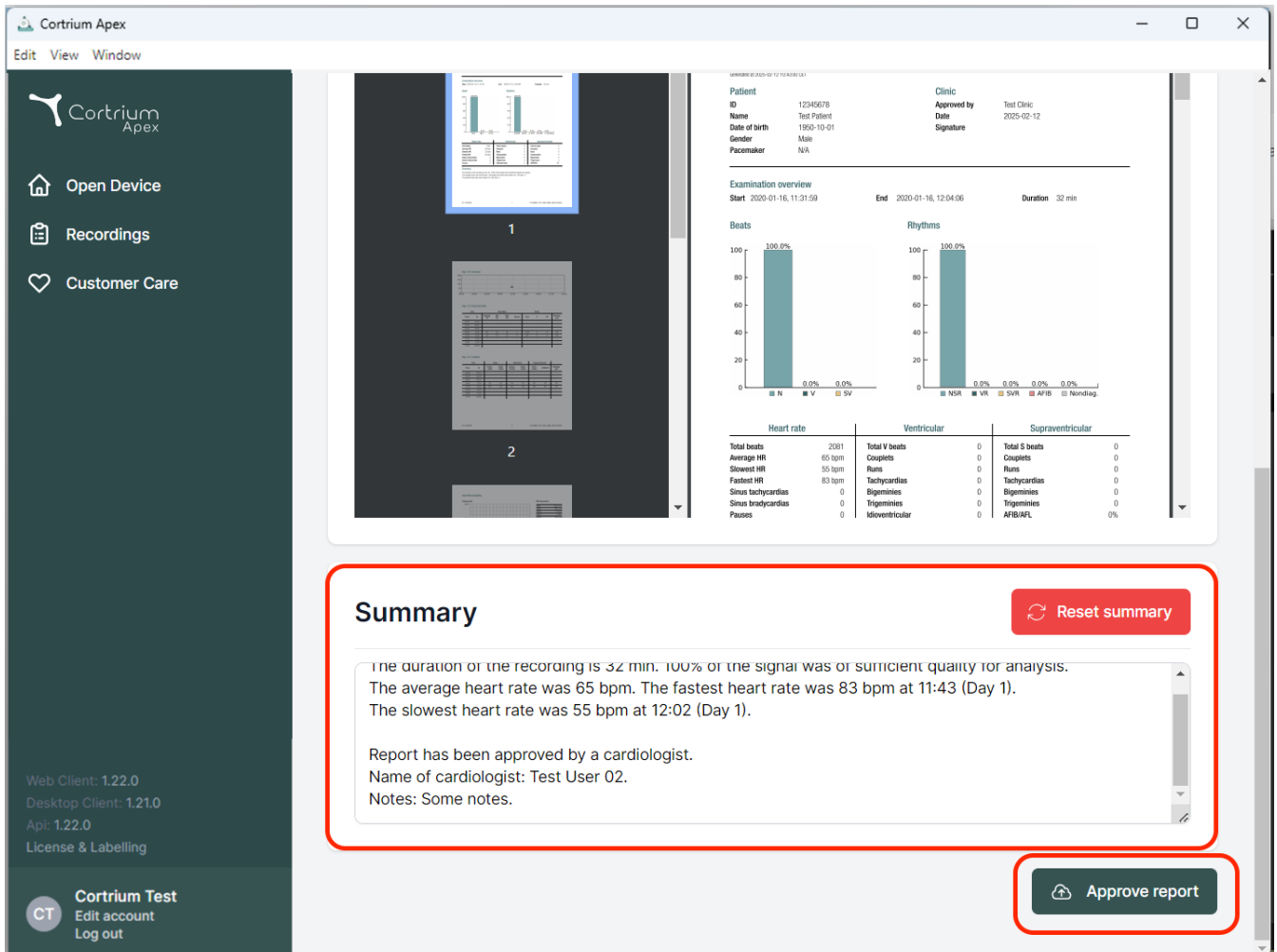The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area shows a patient report for 'Test Patient' (ID: 12345678, Date of birth: 1950-10-01, Gender: Male, Pacemaker: N/A). The report includes an 'Examination overview' section with start/end times and duration. Below this are two bar charts: 'Beats' (showing 100.0% for N, 0.0% for V, 0.0% for SV) and 'Rhythms' (showing 100.0% for NSR, 0.0% for VR, 0.0% for SVR, 0.0% for AFB, 0.0% for Nonrhythm). A table at the bottom provides detailed heart rate and rhythm statistics.

Below the report, a red-bordered box highlights the 'Summary' section. It contains the following text:

Summary Reset summary

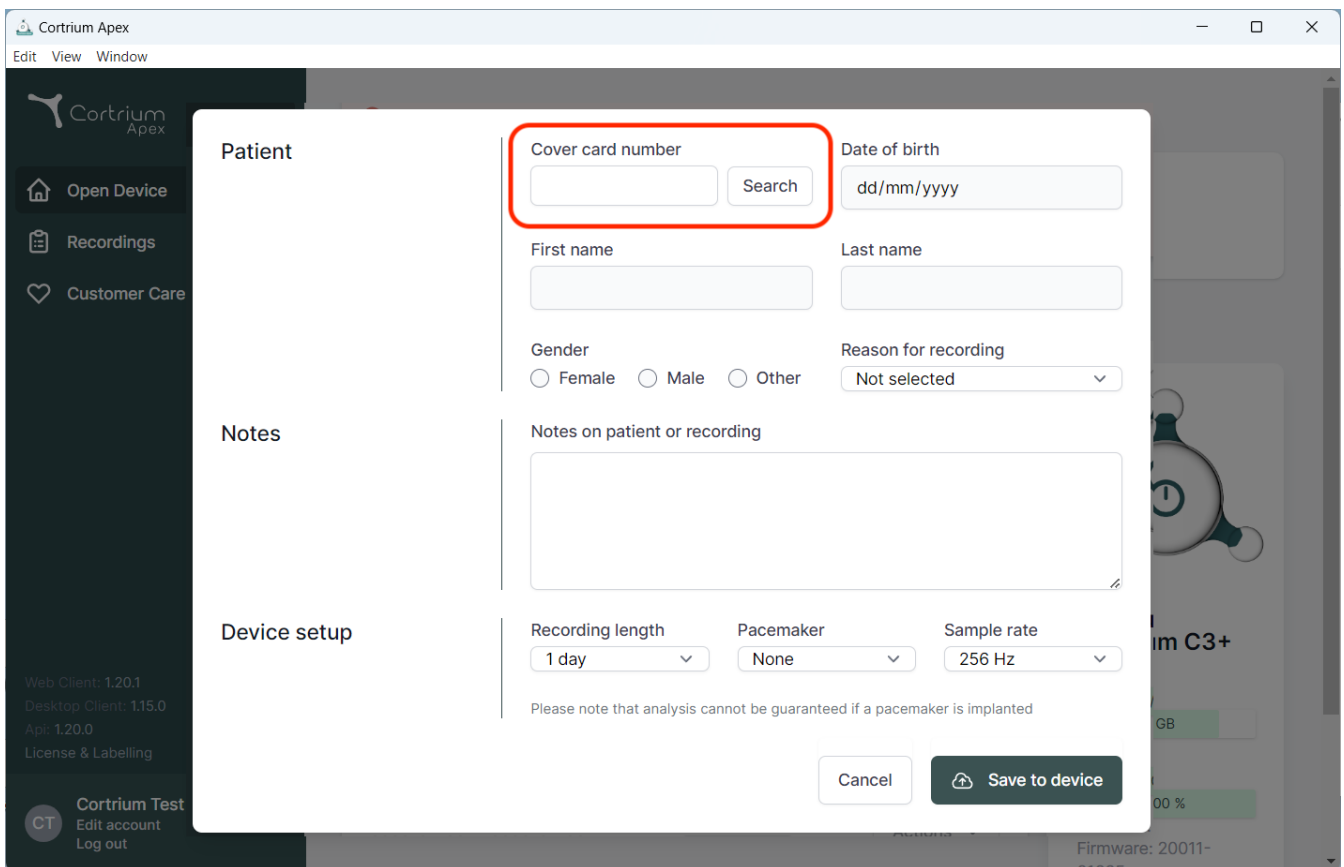
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
 Name of cardiologist: Test User 02.
 Notes: Some notes.

At the bottom right, a red-bordered box highlights the 'Approve report' button.

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, allowing for patient search and configuration. The modal is divided into three main sections: "Patient", "Notes", and "Device setup".

Patient Section:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangle.
- Date of birth:** A text input field with the placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup options, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

The background of the application shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar contains version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0, License & Labelling) and a user profile section for "Cortrium Test" with options to "Edit account" and "Log out".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark
E-mail : info@cortrium.com • Site Web : www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
48 / 48