Cortrium Apex 1.27

Cortrium Apex 1.27

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
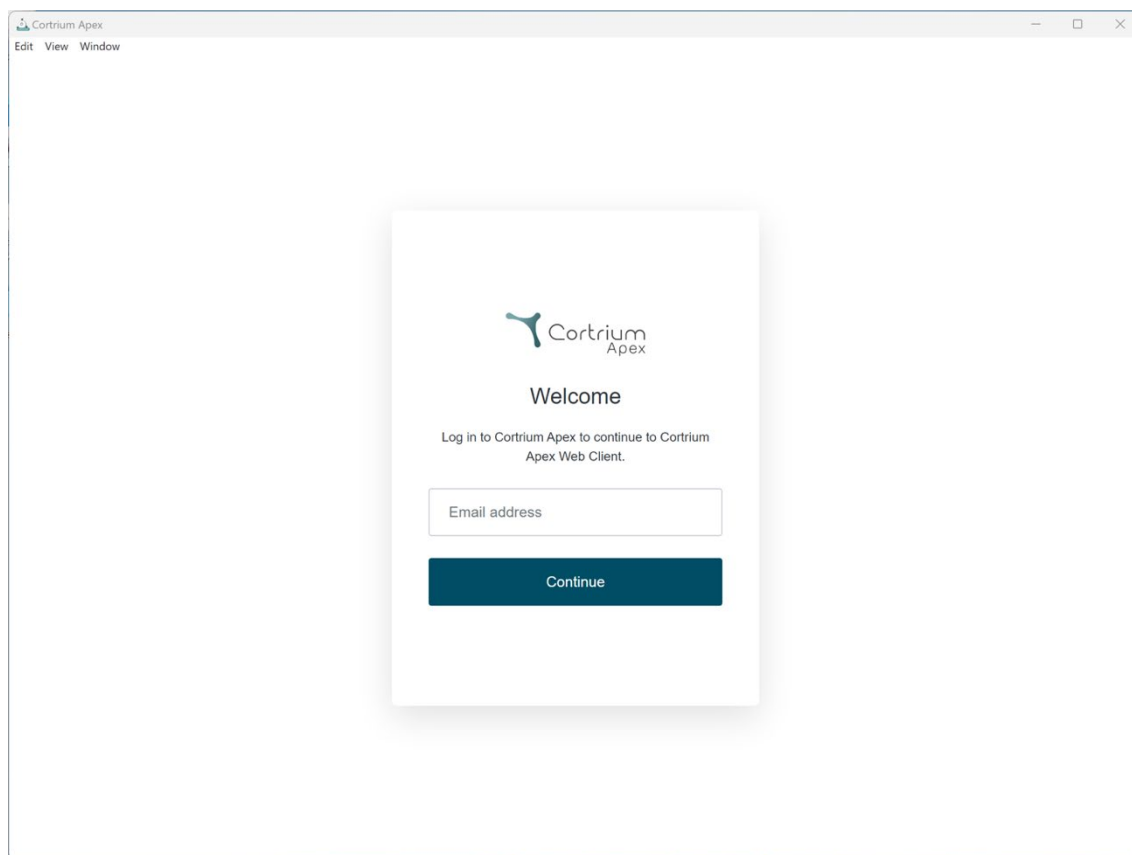
The screenshot shows a web browser window titled "Cortrium Apex". The browser's menu bar includes "Edit", "View", and "Window". In the center of the browser window is a white login card with a subtle shadow. The card features the Cortrium Apex logo at the top, followed by the heading "Welcome". Below the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." There is a text input field labeled "Email address" and a dark blue button labeled "Continue" below it.

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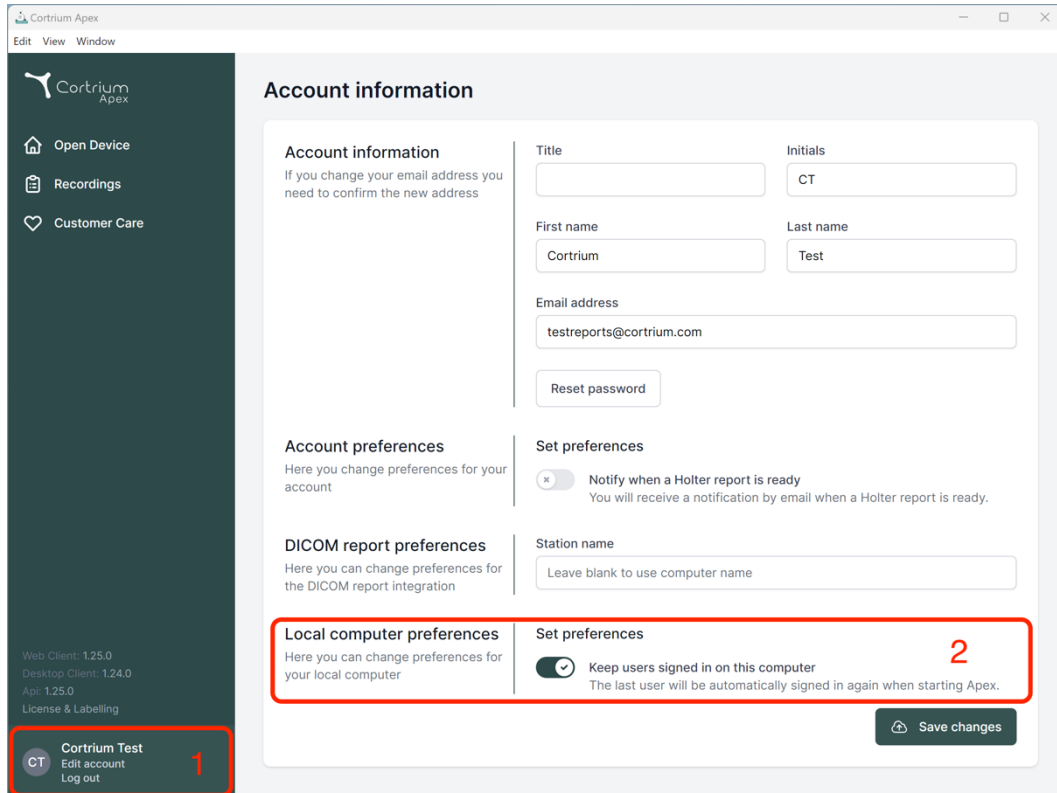The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details for Web Client, Desktop Client, and API. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with a red box labeled '1' around it. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' for changing account settings; 'DICOM report preferences' for integration settings; and 'Local computer preferences' for local device settings. The 'Local computer preferences' section has a red box labeled '2' around it, highlighting the 'Keep users signed in on this computer' toggle, which is currently turned on. A 'Save changes' button is located at the bottom right of the main content area.

Account information

If you change your email address you need to confirm the new address

Title: Initials:

First name: Last name:

Email address:

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name

Local computer preferences

Here you can change preferences for your local computer

Set preferences

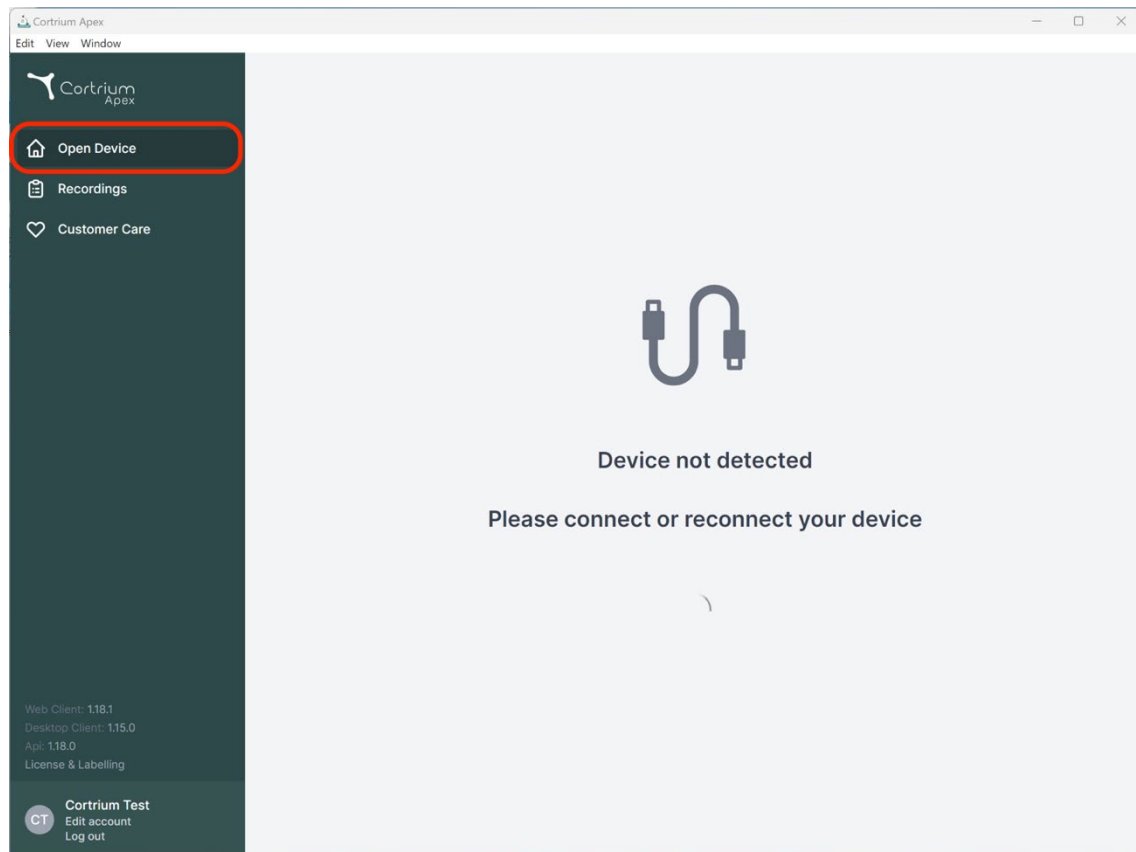
☒ Keep users signed in on this computer
The last user will be automatically signed in again when starting Apex.

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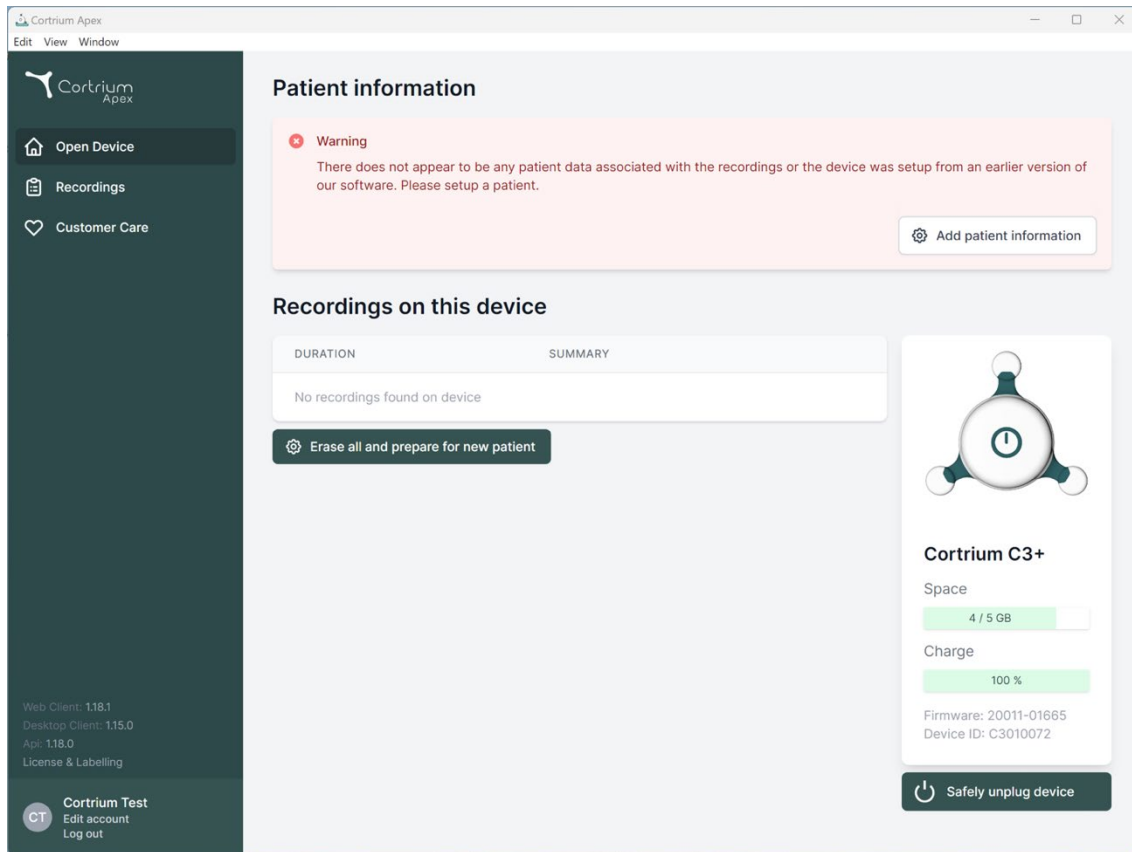
Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Save to device

Warning This will erase all recordings on the device

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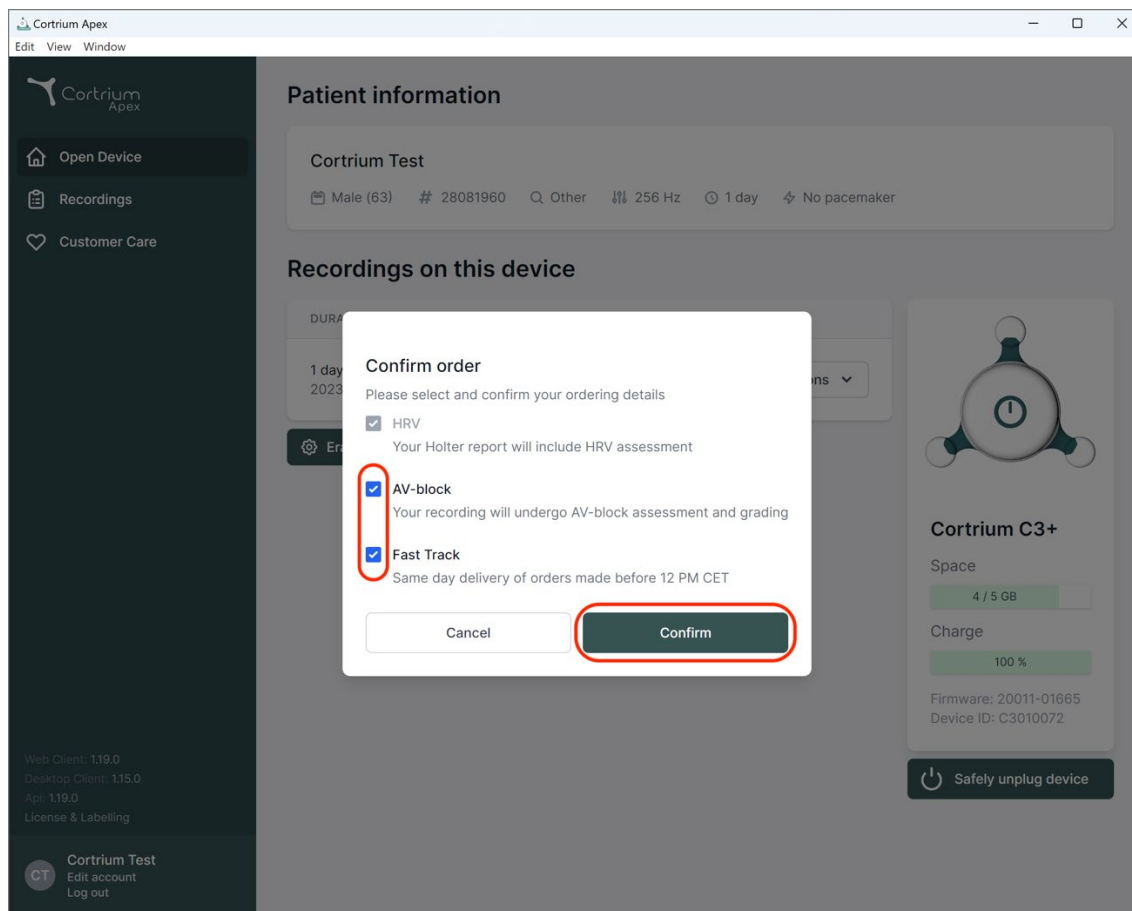
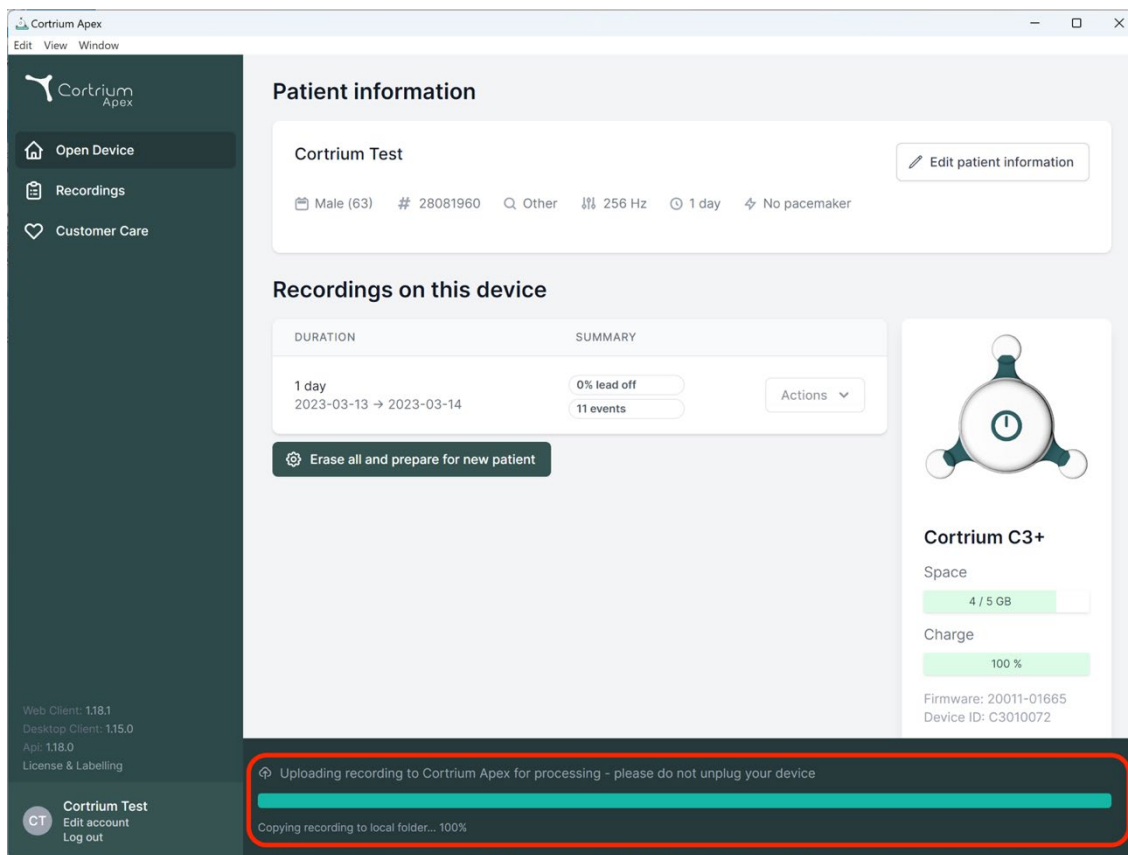
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is divided into sections: Patient information, Recordings on this device, and device status for Cortrium C3+.

Patient information

Cortrium Test

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

[Edit patient information](#)

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

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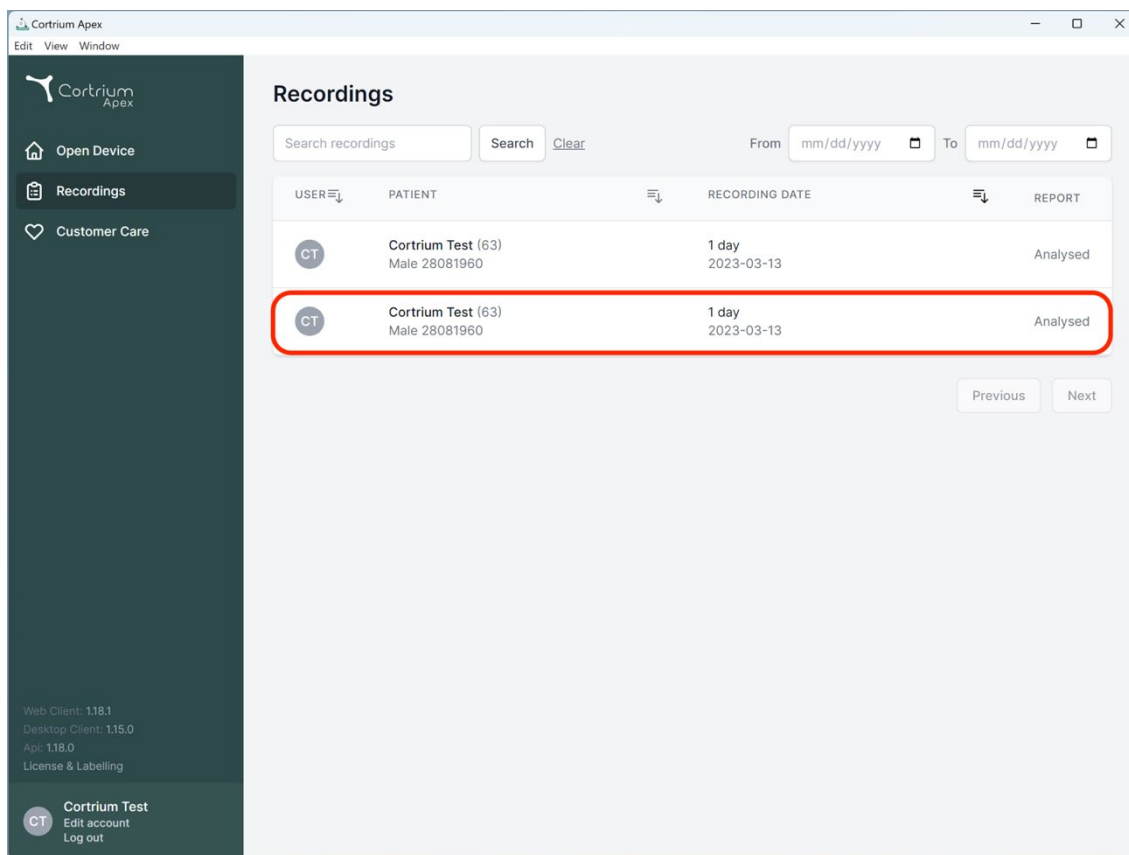
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area shows a table of recordings with the following columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains two rows of data, with the second row highlighted by a red rectangular border. Below the table are 'Previous' and 'Next' navigation buttons. The bottom of the sidebar contains version information and a user profile section for 'Cortrium Test'.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

Cortrium Apex

Edit View Window

[Open Device](#)

[Recordings](#)

[Customer Care](#)

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT

Cortrium Test
Edit account
Log out

Recording details

Delete recording

Cortrium Test

Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off

2023-03-13 11:54 256 Hz C3100175

View recording

View >

Download recording in .EDF format

Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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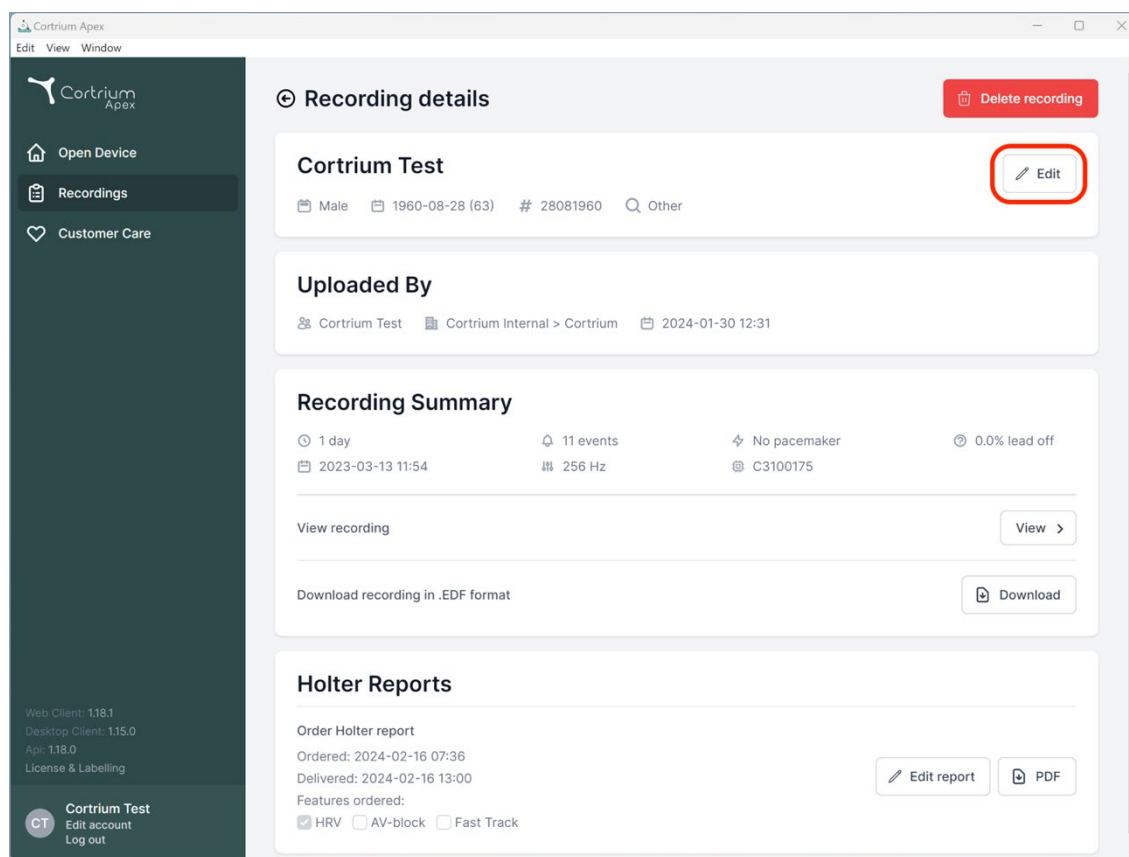
2023-03-13 11:54 256 Hz C3100175

View recording View

Download recording in .EDF format Download

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
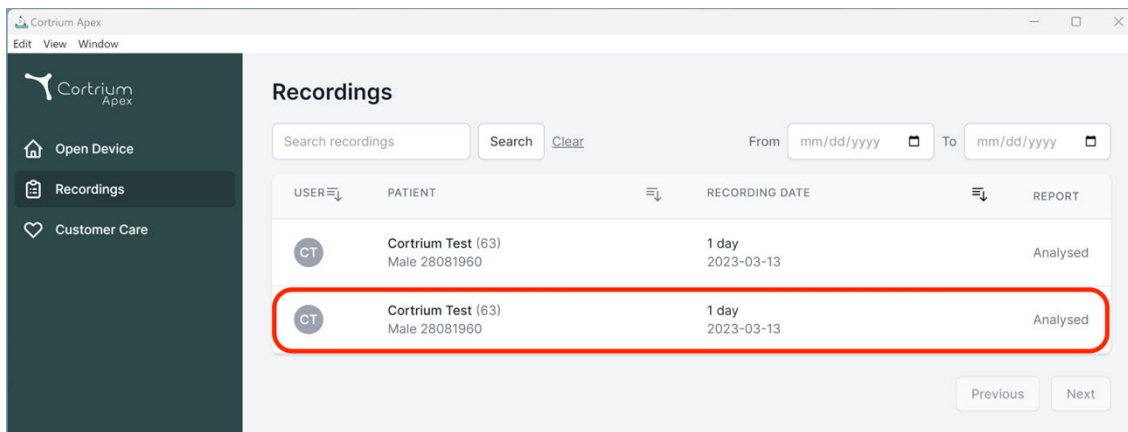
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. It features a search bar and a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains two entries for 'Cortrium Test (63)' on '2023-03-13', both marked as 'Analysed'. The second entry is highlighted with a red rectangle. Navigation buttons 'Previous' and 'Next' are at the bottom right.

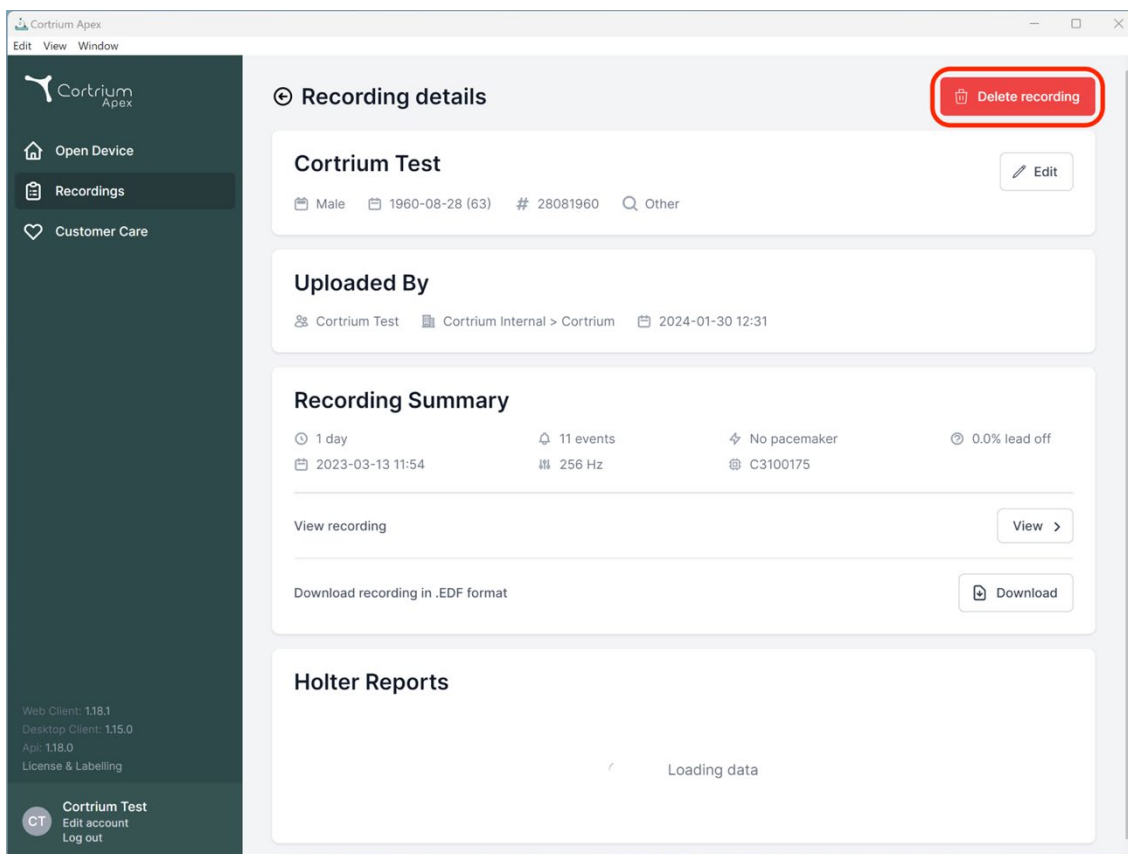The screenshot shows the 'Recording details' page for 'Cortrium Test'. It includes patient information (Male, 1960-08-28 (63), # 28081960), upload details (Uploaded By: Cortrium Test, 2024-01-30 12:31), and a recording summary (1 day, 11 events, No pacemaker, 0.0% lead off). A red rectangle highlights the 'Delete recording' button in the top right corner. The bottom section is labeled 'Holter Reports' and shows 'Loading data'.

Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

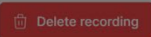
 Recordings

 Customer Care

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

 Cortrium Test
Edit account
Log out

Recording details

Cortrium Test 

 Male  1960-08-28 (63)  # 28081960  Other

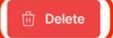
Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete


You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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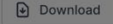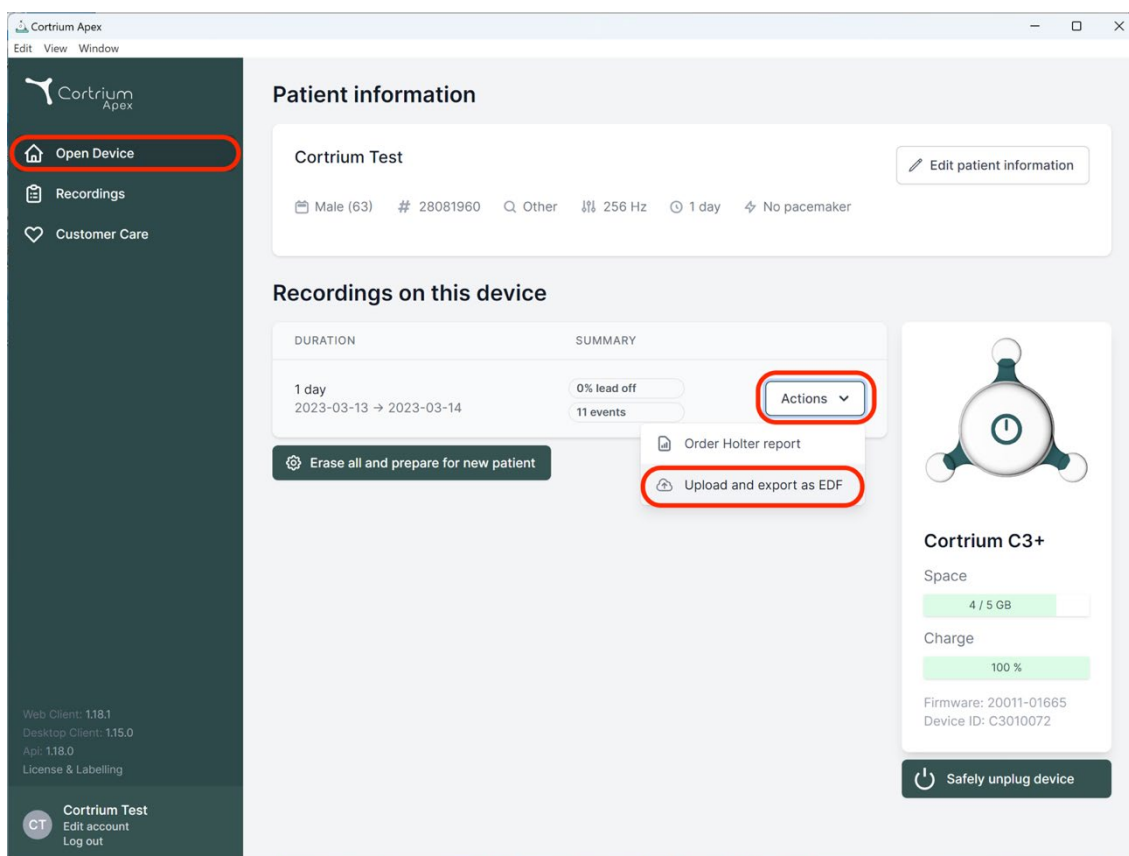
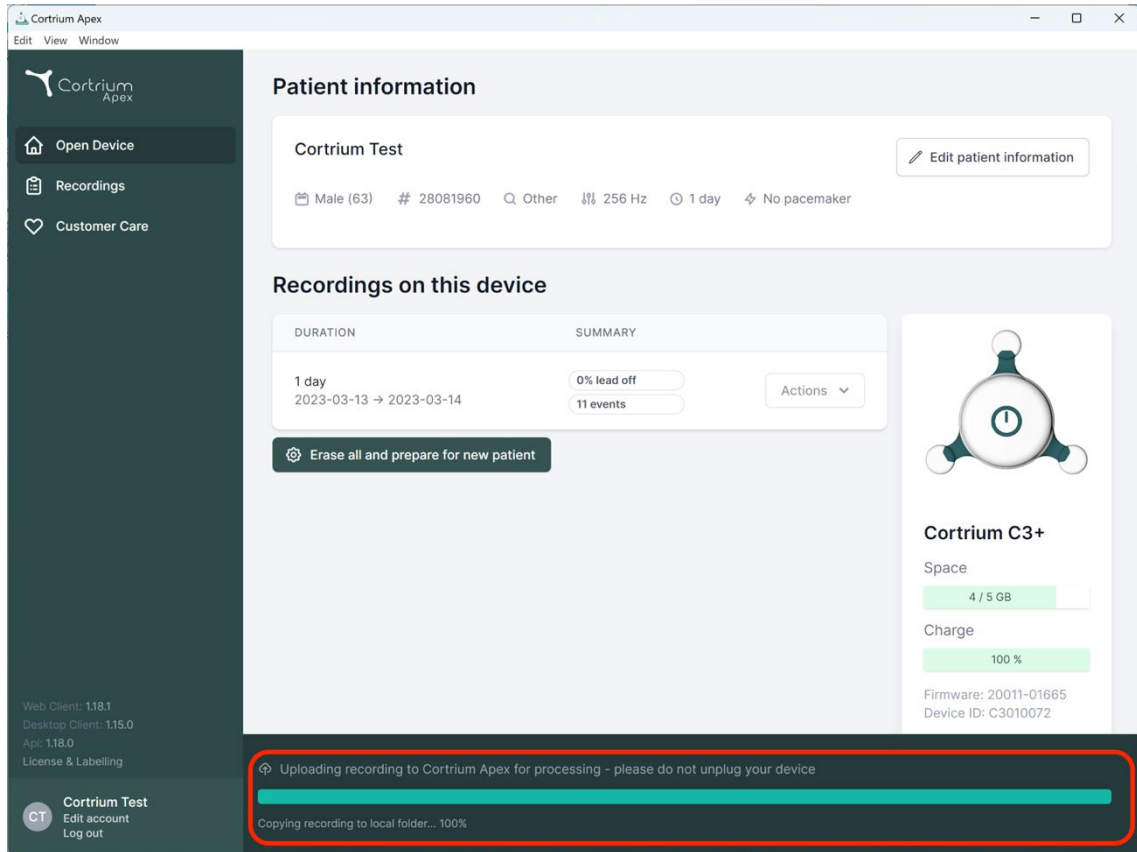
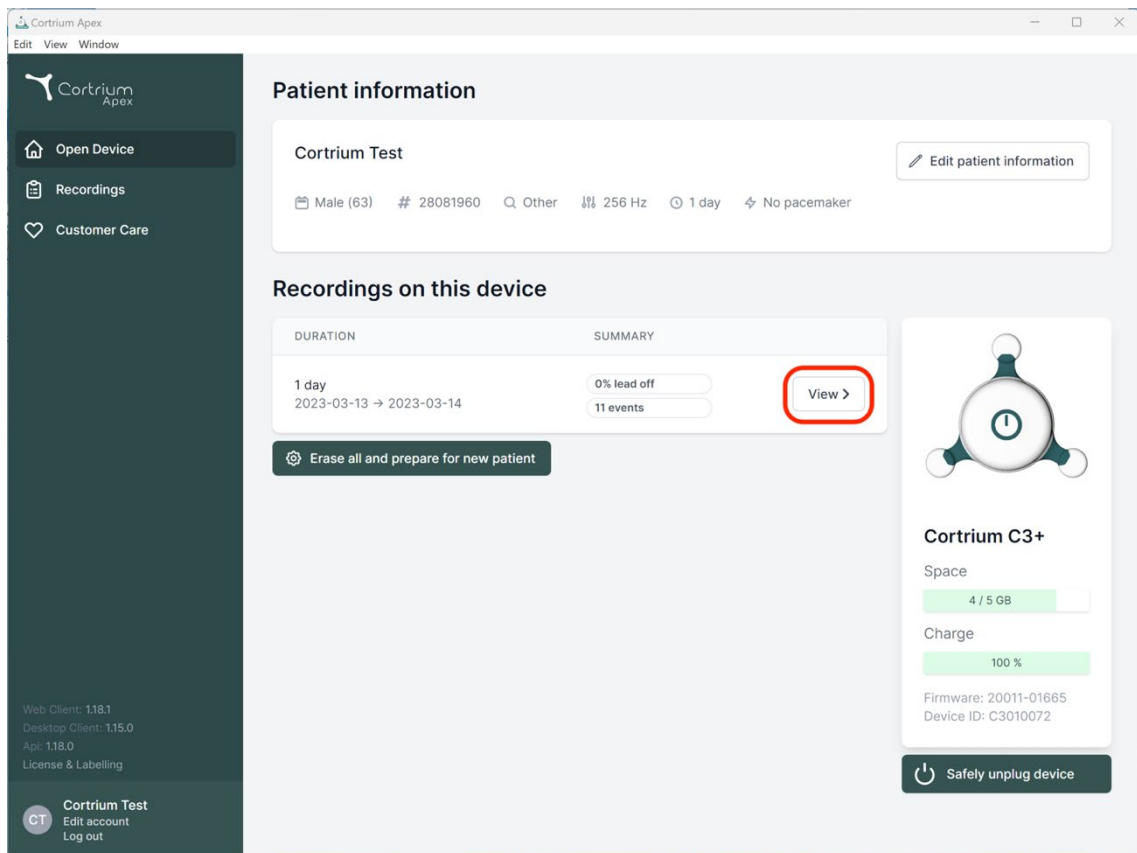
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

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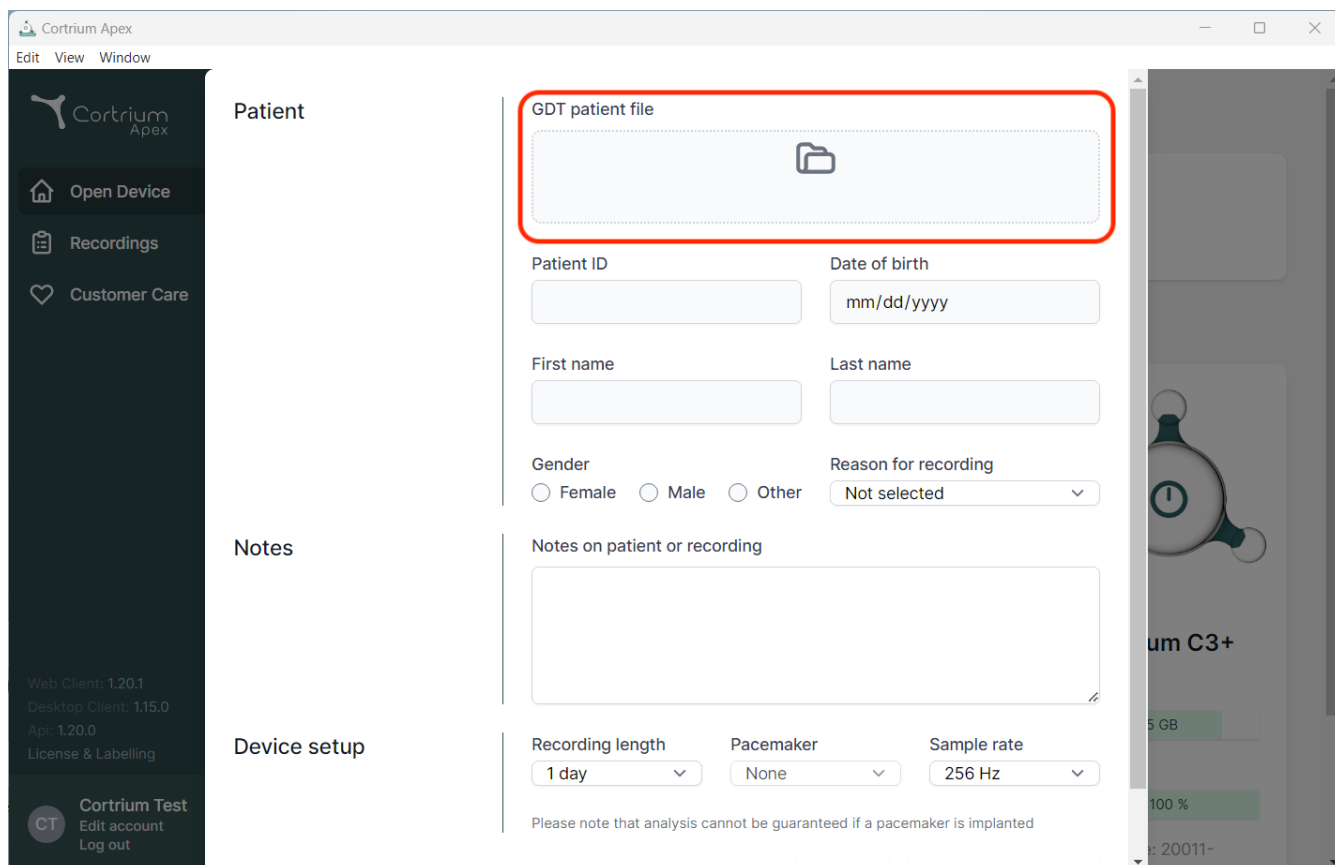
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", and "Api: 1.20.0". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

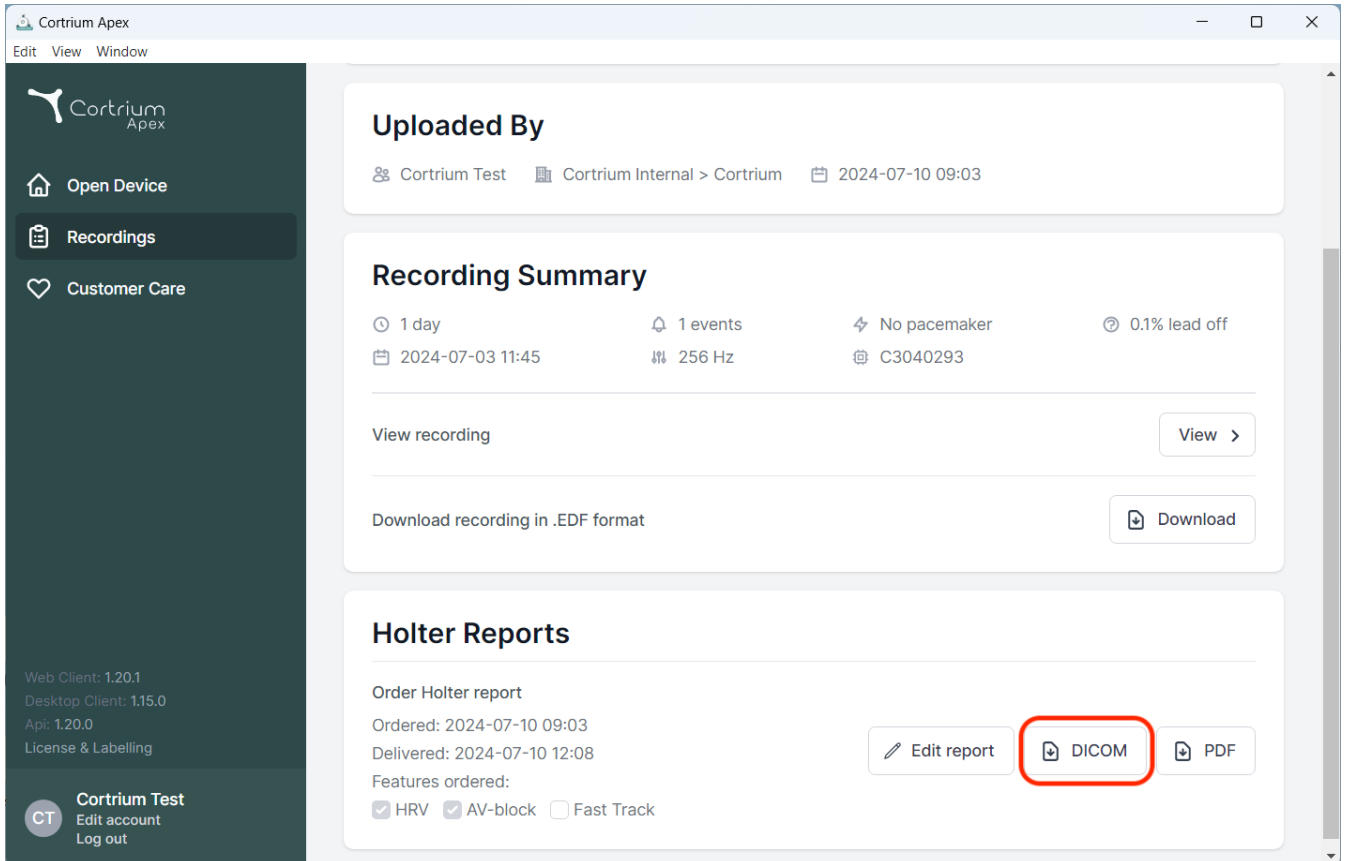
The main content area is divided into three sections: "Patient", "Notes", and "Device setup".

- Patient section:** Contains a "GDT patient file" upload area (highlighted with a red box) with a folder icon. Below it are input fields for "Patient ID", "Date of birth" (format: mm/dd/yyyy), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown for "Reason for recording" (currently "Not selected").
- Notes section:** Contains a text area labeled "Notes on patient or recording".
- Device setup section:** Contains dropdowns for "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). Below these is a disclaimer: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device screen is visible, showing a heart icon and the text "um C3+".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' sampling rate, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (for .EDF format).
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists ordered features: 'HRV' (checked), 'AV-block' (checked), and 'Fast Track' (unchecked). At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

The bottom left of the sidebar shows version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and a user profile for 'Cortrium Test' with 'Edit account' and 'Log out' options.

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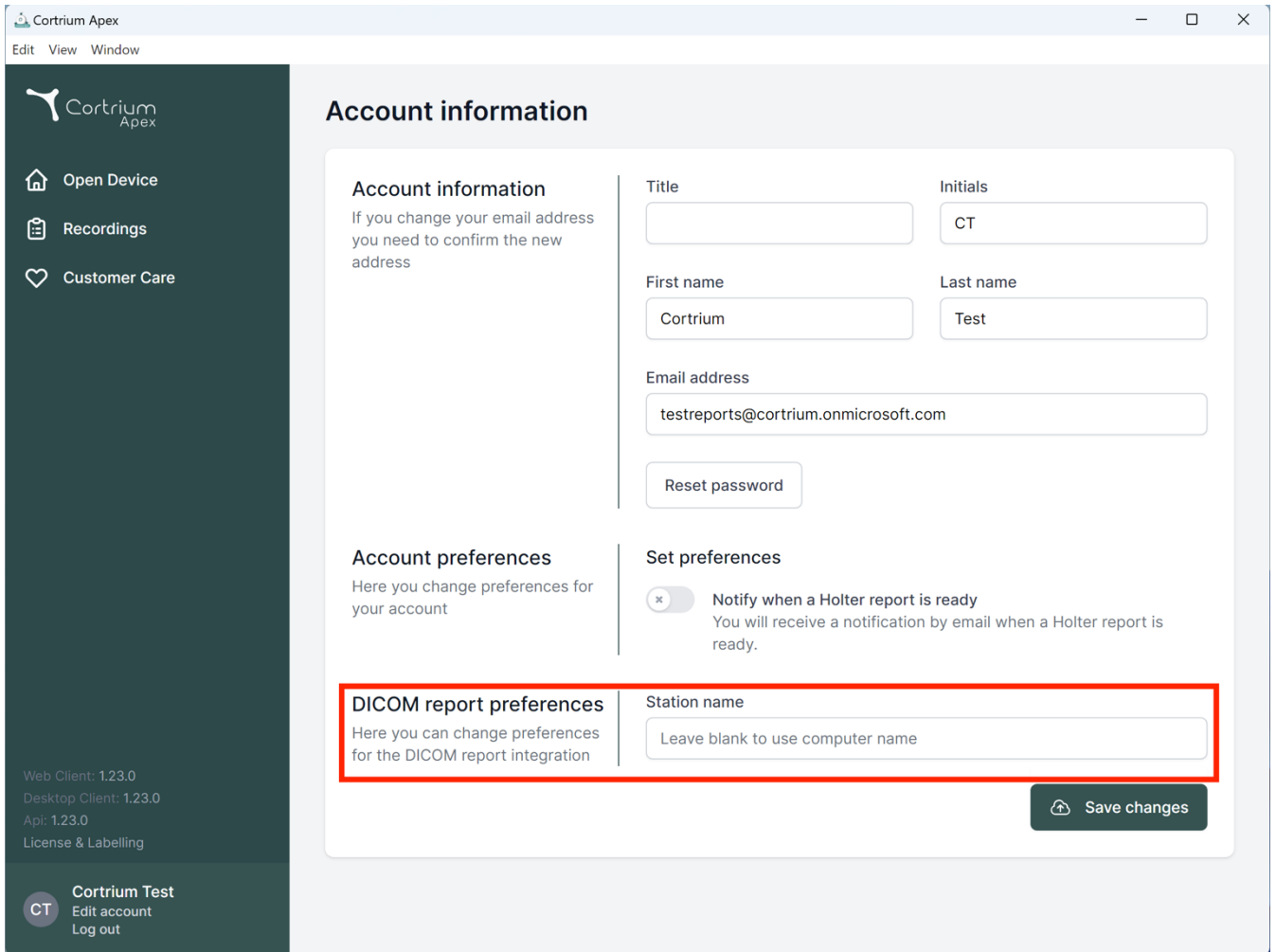Edit View Window

Account information

Account information
If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name:

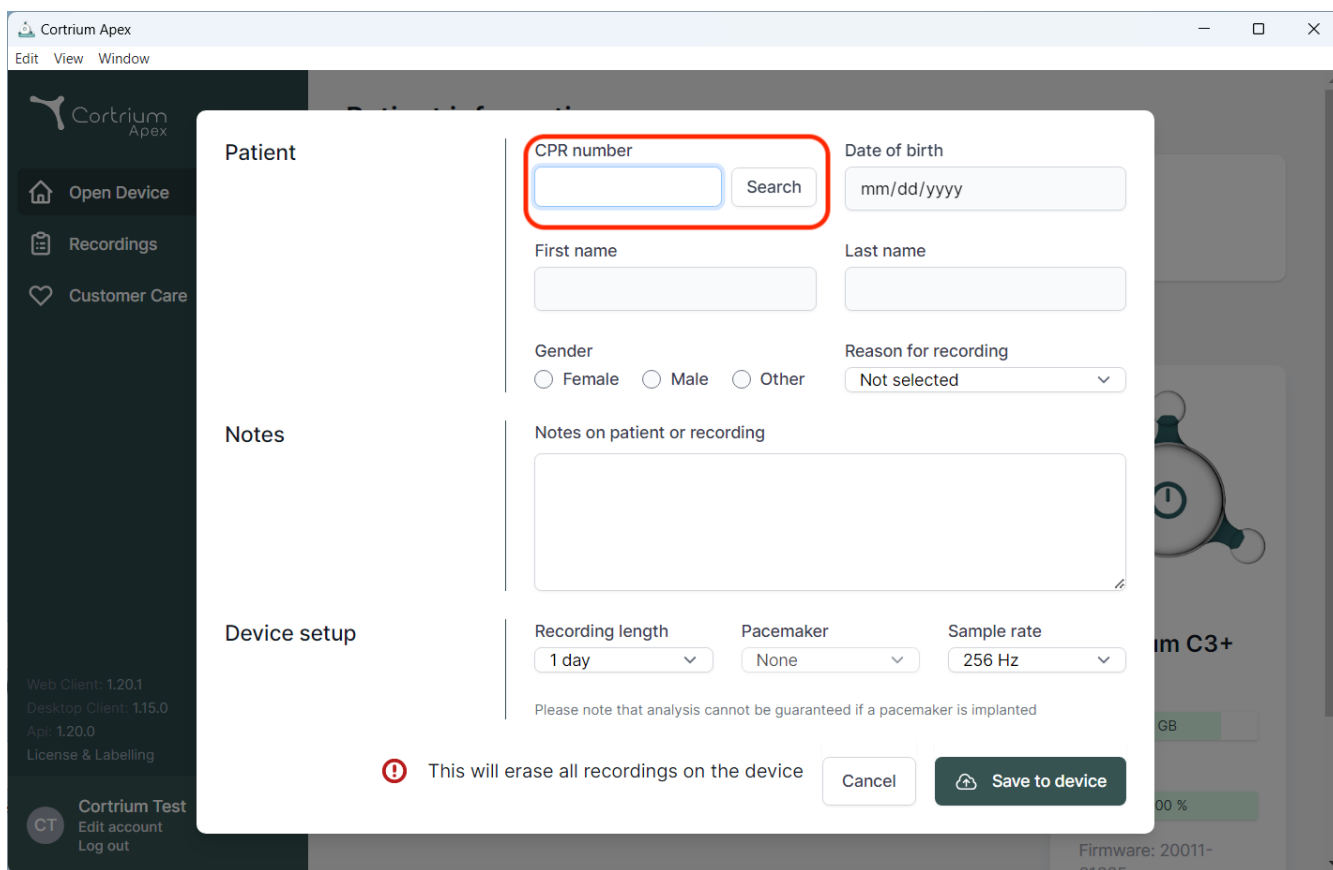
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a search form. The form includes the following fields and controls:

- CPR number:** A text input field with a blue border, highlighted by a red rectangle, and a "Search" button next to it.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Three radio buttons labeled "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup fields, a warning message states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom of the modal, there is a red warning icon and the text "This will erase all recordings on the device", followed by "Cancel" and "Save to device" buttons.

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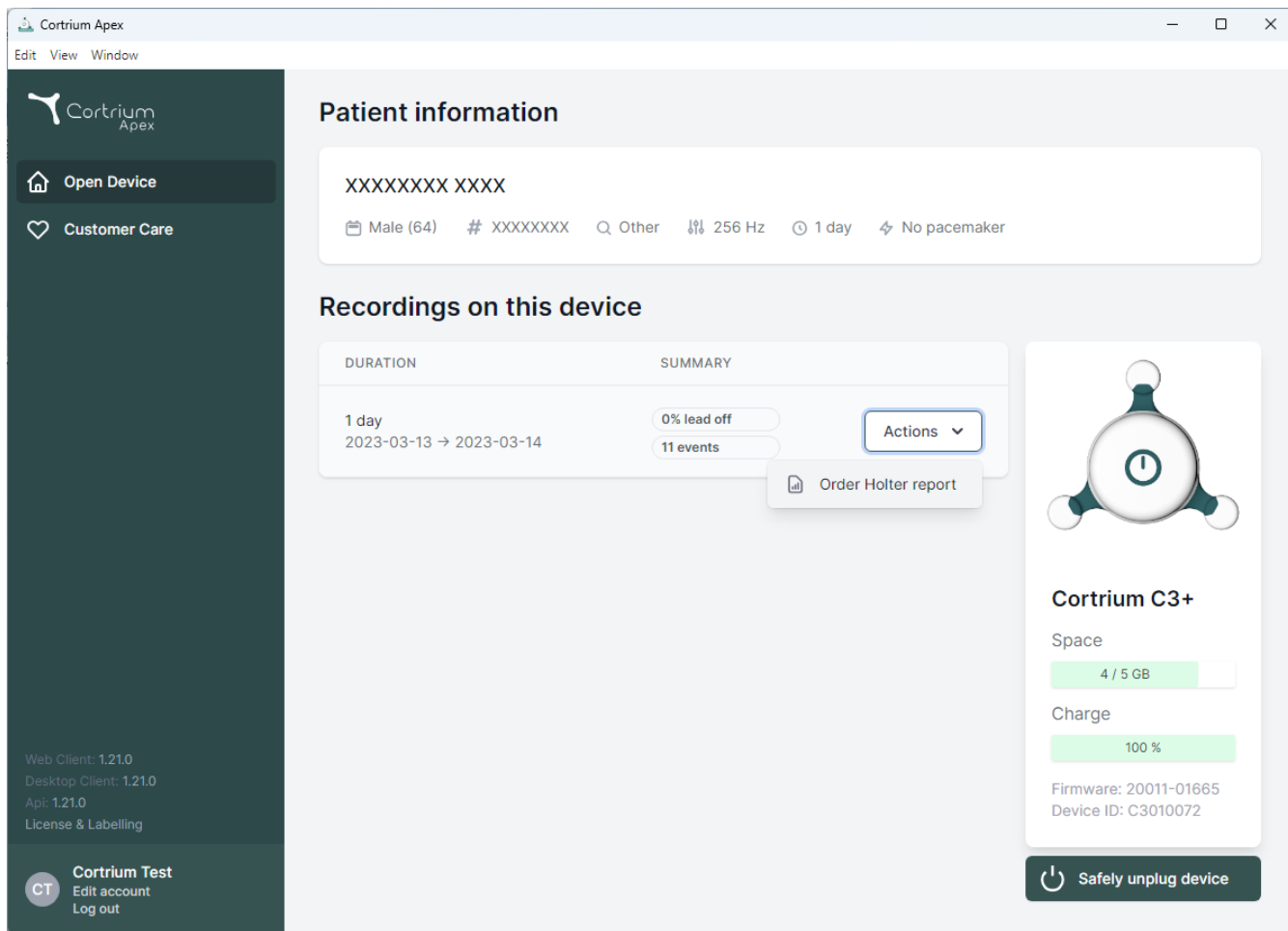The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device', 'Customer Care', and version information (Web Client: 1.21.0, Desktop Client: 1.21.0, Api: 1.21.0, License & Labelling). The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

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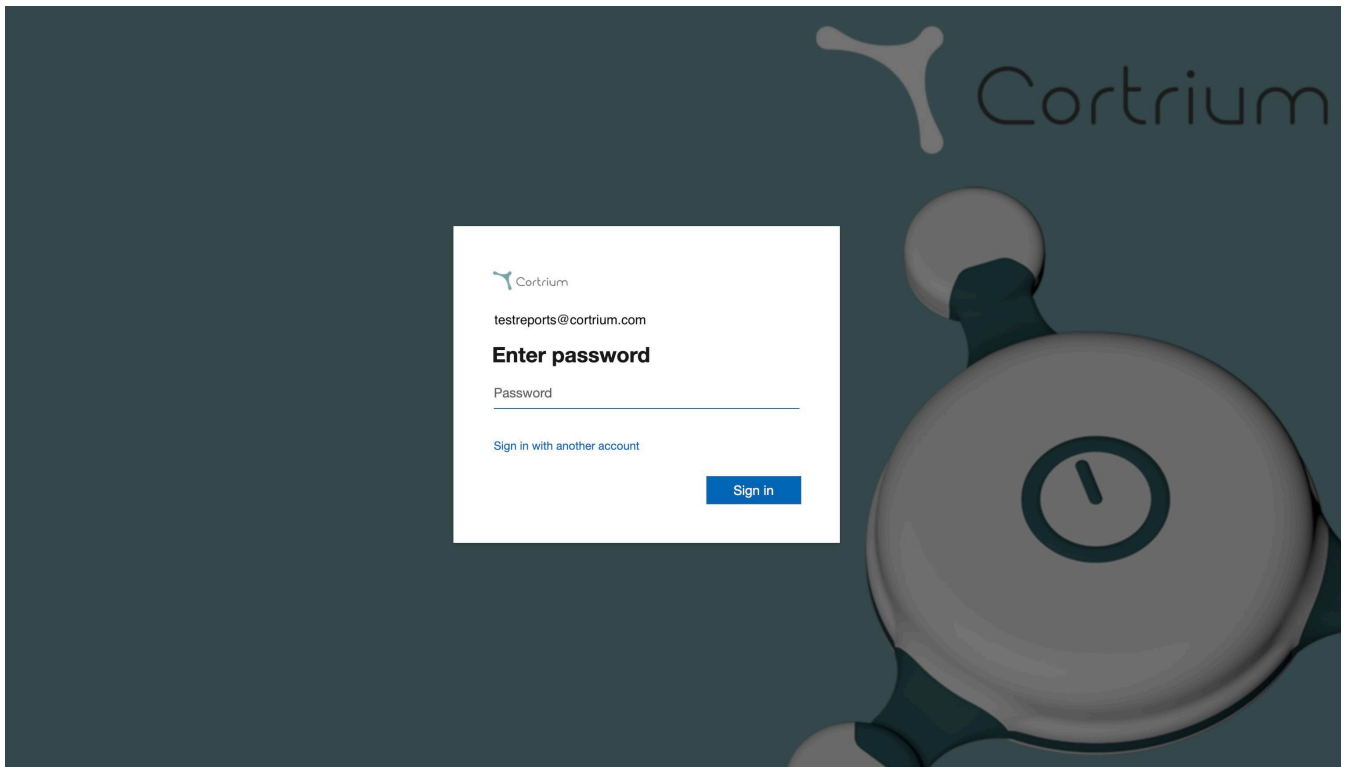
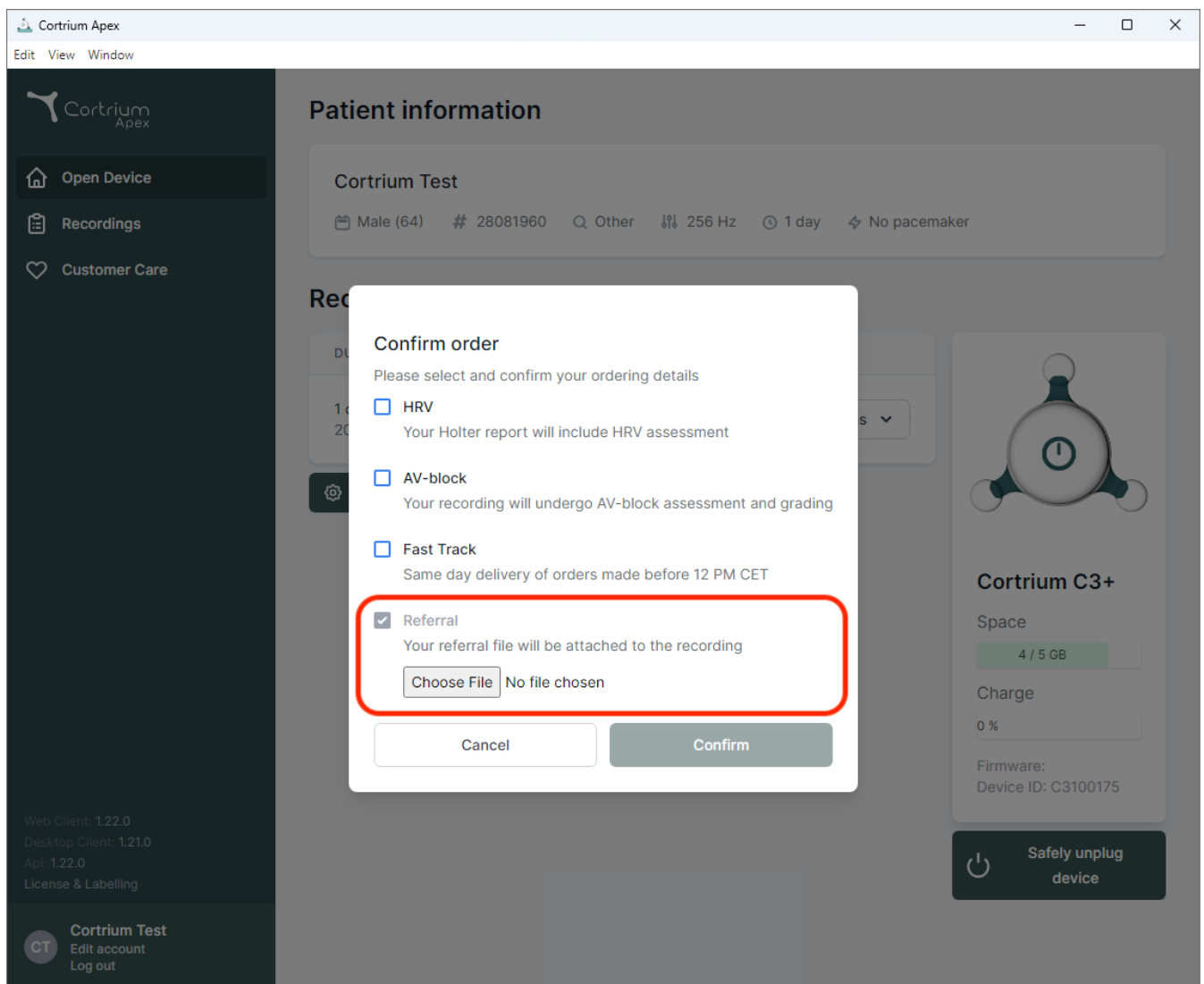
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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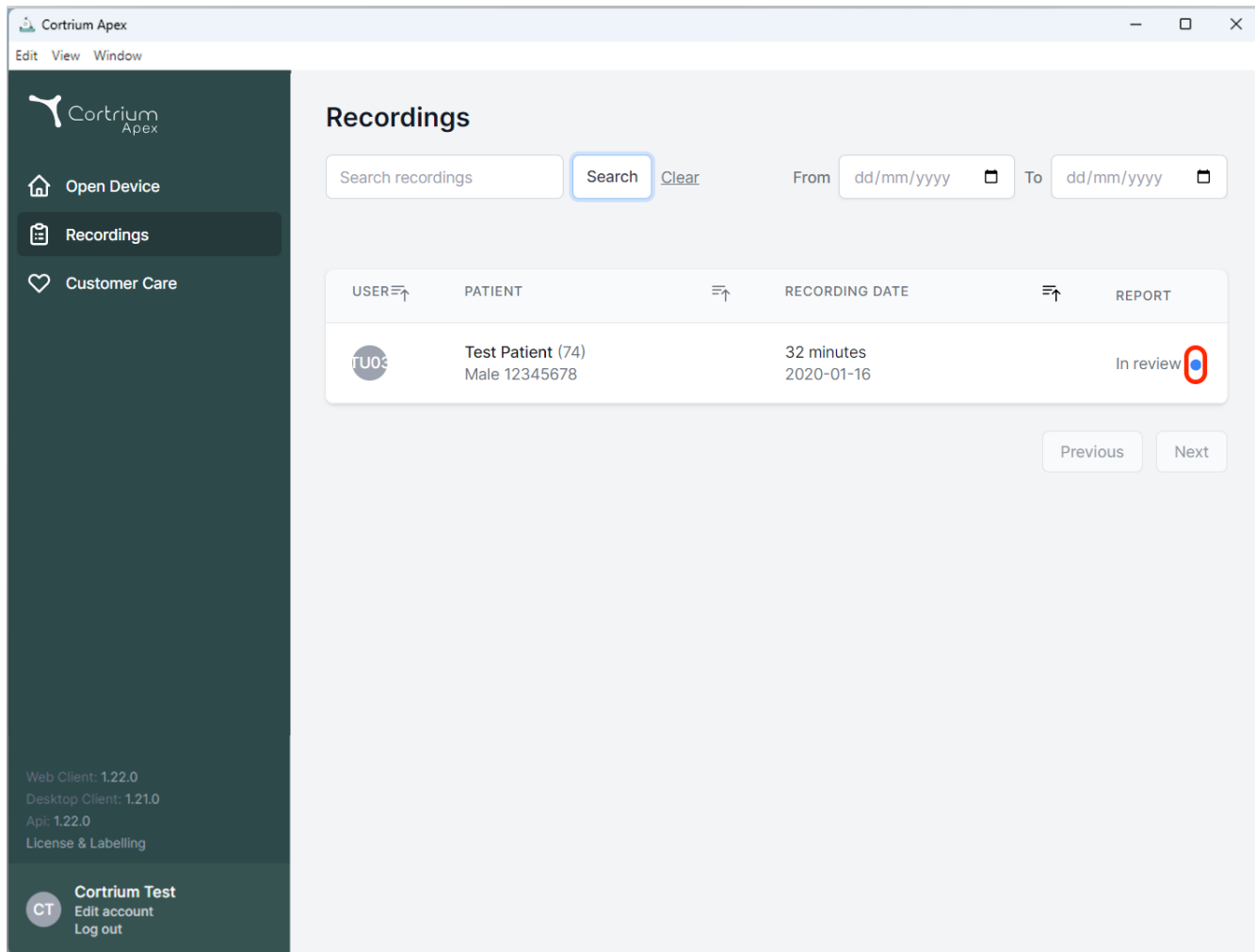
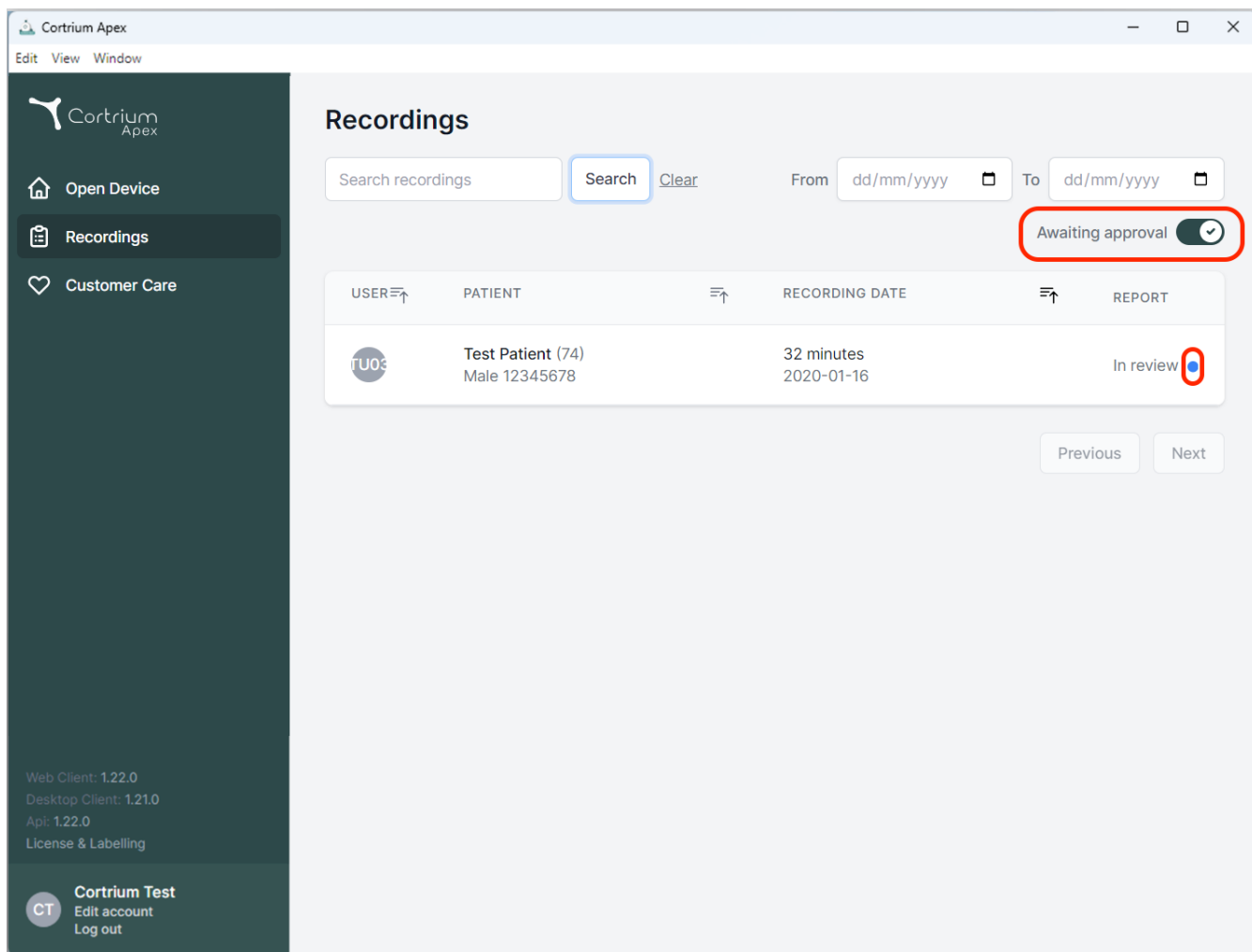
Edit View Window

Recordings

Search recordings Search Clear From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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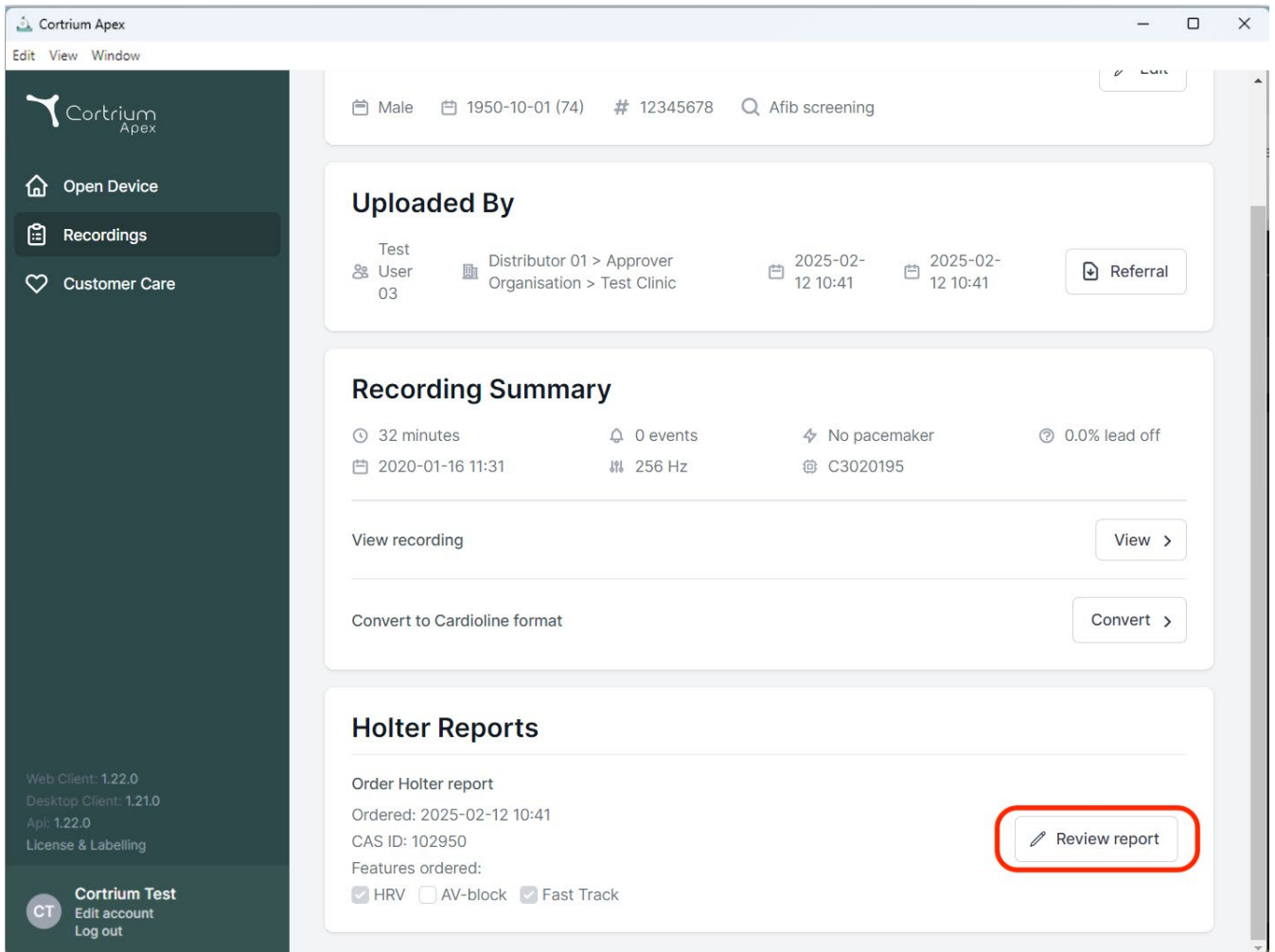
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo and three main menu items: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into several sections:

- Header Information:** Patient details including 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'.
- Uploaded By:** A section showing the test user (User 03), distributor (Distributor 01 > Approver Organisation > Test Clinic), and upload dates (2025-02-12 10:41). A 'Referral' button is present.
- Recording Summary:** A section providing key recording metrics: 32 minutes duration, 0 events, 256 Hz sampling rate, and 0.0% lead off. It also includes a 'View recording' button and a 'Convert to Cardioline format' button.
- Holter Reports:** A section for managing holter reports, showing the order date (2025-02-12 10:41) and CAS ID (102950). A 'Review report' button is highlighted with a red rectangle.

The bottom of the sidebar contains version information (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0) and a 'Cortrium Test' section with links for 'Edit account' and 'Log out'.

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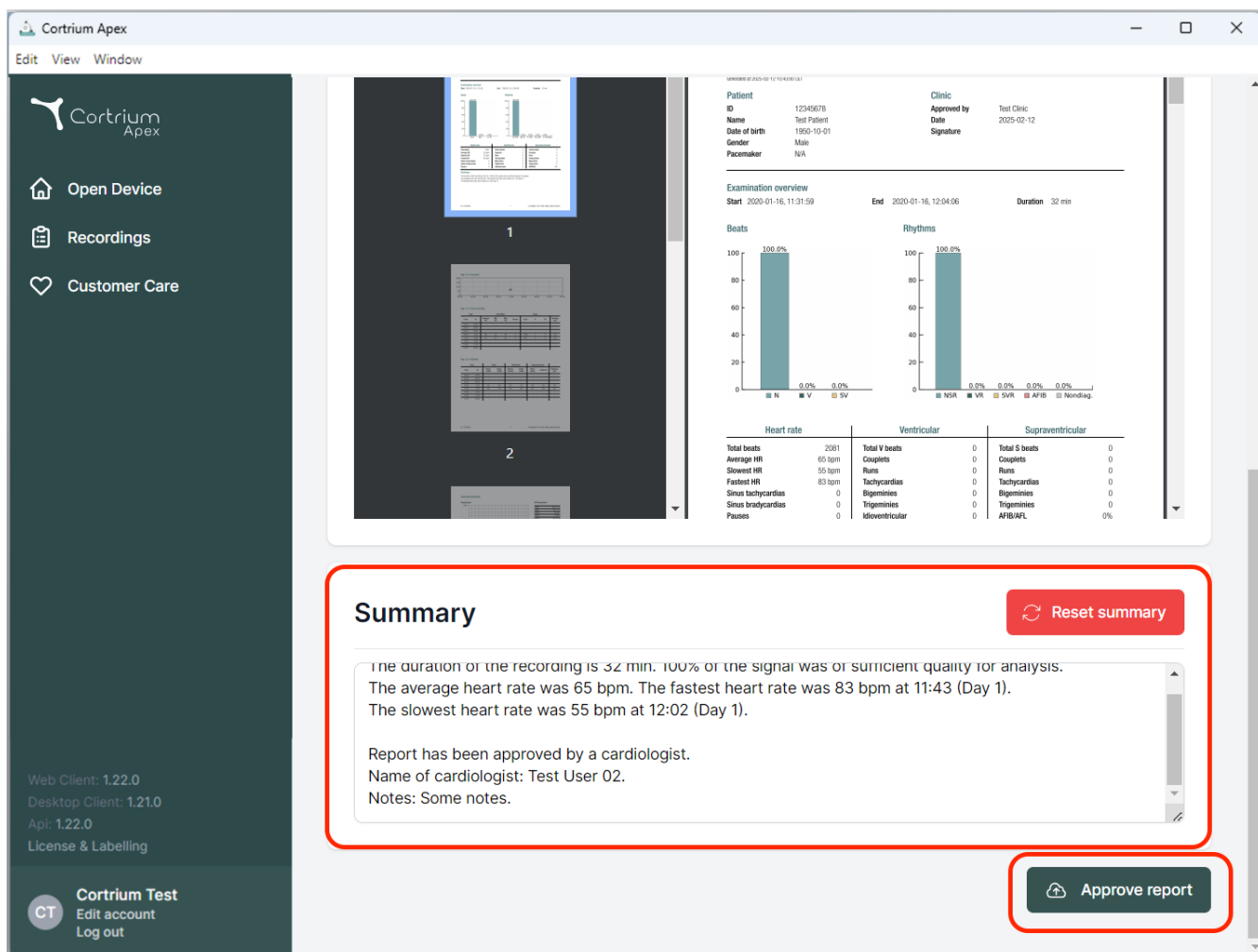Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Apl: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

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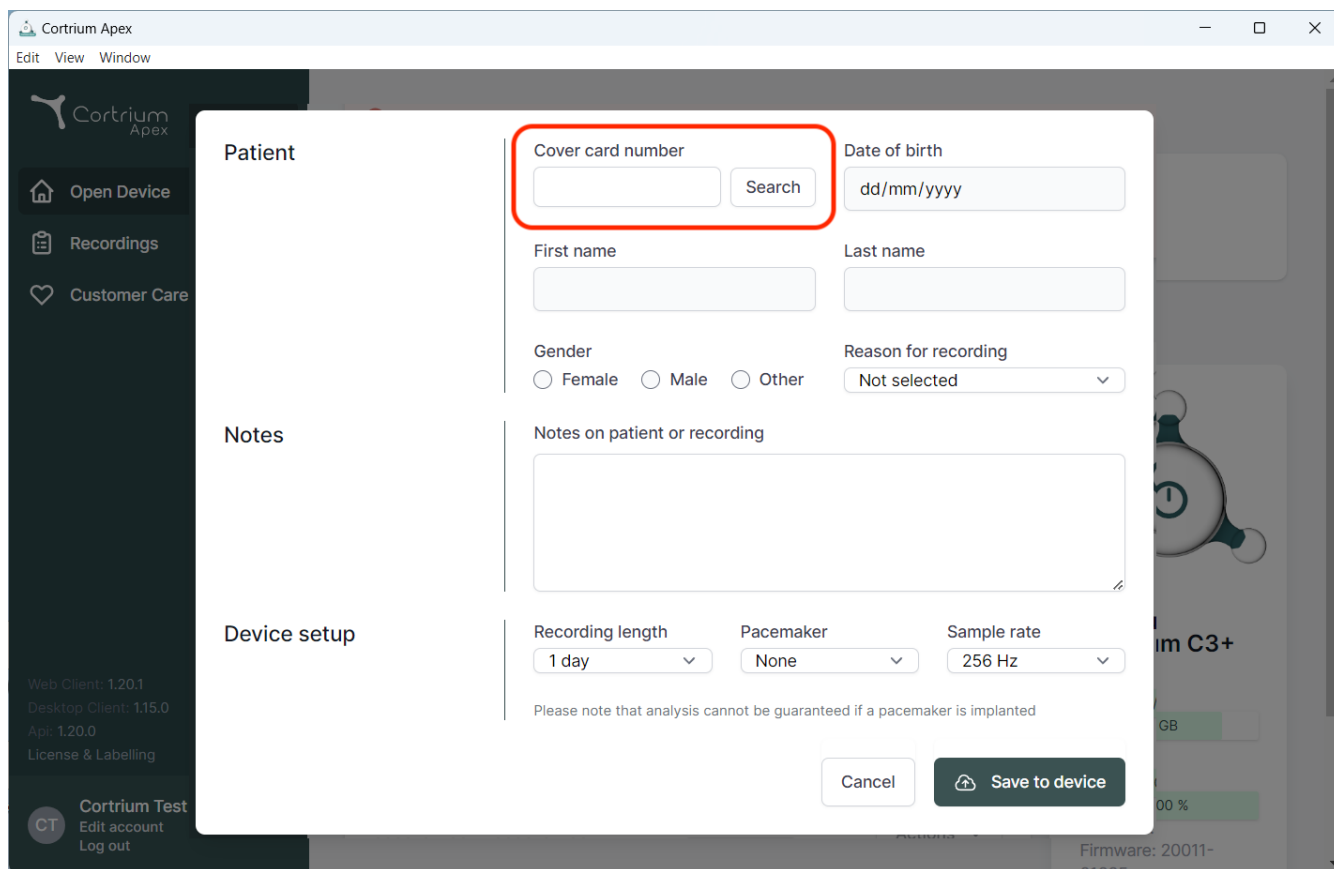
The screenshot displays the Cortrium Apex software interface. A modal window titled "Patient" is open, allowing for patient data entry. The form is organized into sections: "Patient", "Notes", and "Device setup".

Patient Section:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup options, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

The background interface shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom left corner displays version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right corner shows "Cortrium Test", "Edit account", "Log out", and "Firmware: 20011-0100".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dánia
E-mail: info@cortrium.com • Weboldal: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47