Cortrium Apex 1.27

Cortrium Apex 1.27

Istruzioni d'uso

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
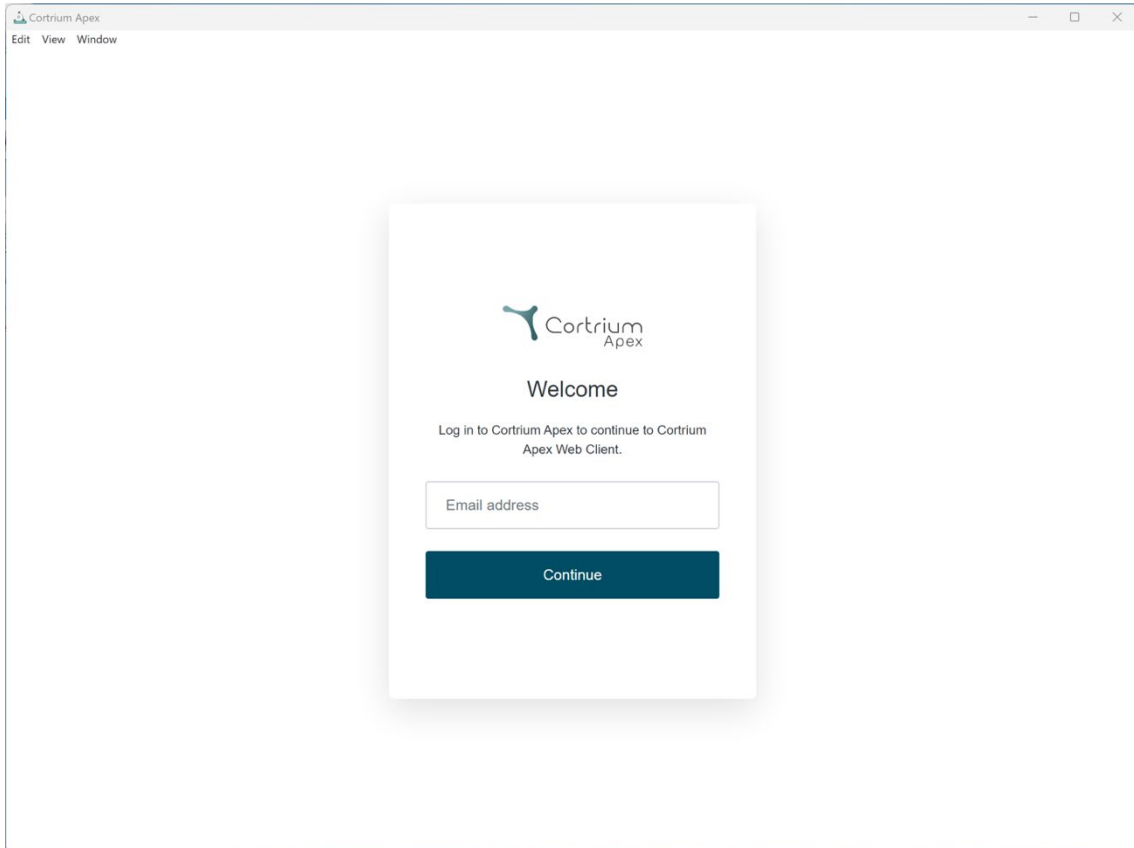
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". The main content area displays a login form centered on the page. The form has a white background and a subtle drop shadow. At the top of the form is the Cortrium Apex logo. Below the logo is the heading "Welcome". Underneath the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." followed by a text input field labeled "Email address". At the bottom of the form is a dark blue button with the text "Continue" in white.

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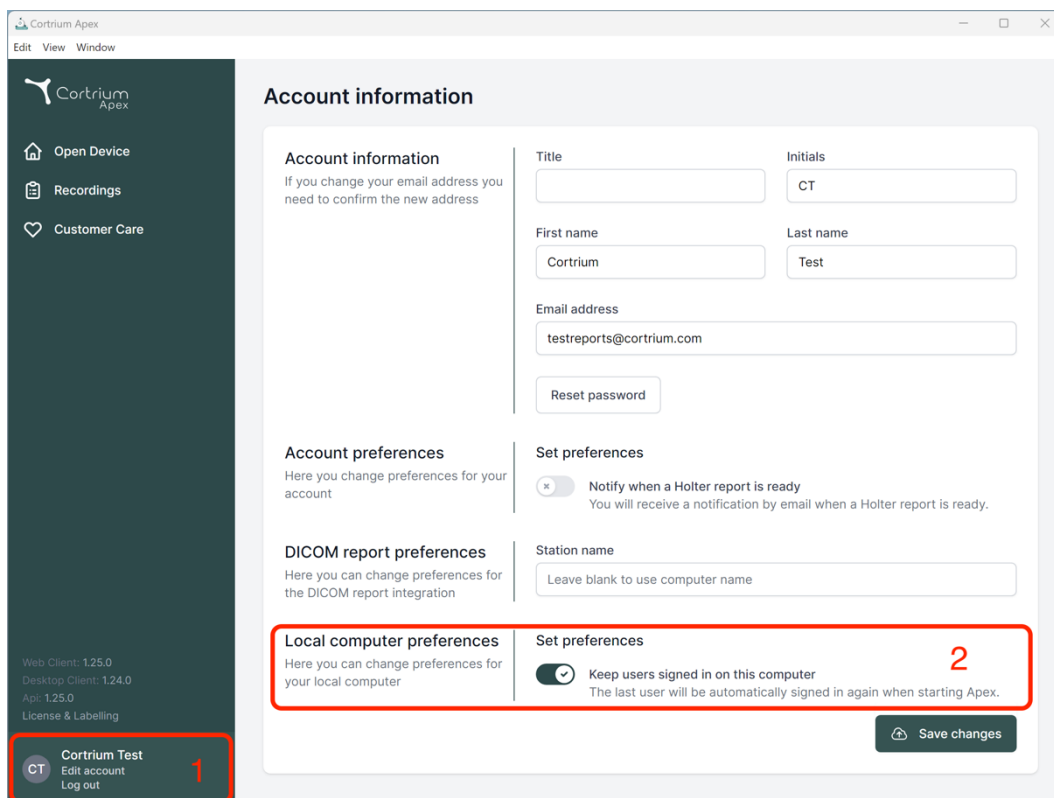
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.25.0, Desktop Client: 1.24.0, Api: 1.25.0) and a 'License & Labelling' link. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with options to 'Edit account' and 'Log out', marked with a red '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' (with fields for Title, Initials, First name, Last name, and Email address, plus a 'Reset password' button), 'Account preferences', 'DICOM report preferences', 'Set preferences' (with a toggle for 'Notify when a Holter report is ready'), and 'Local computer preferences' (with a toggle for 'Keep users signed in on this computer', marked with a red '2'). A 'Save changes' button is located at the bottom right of the main content area.

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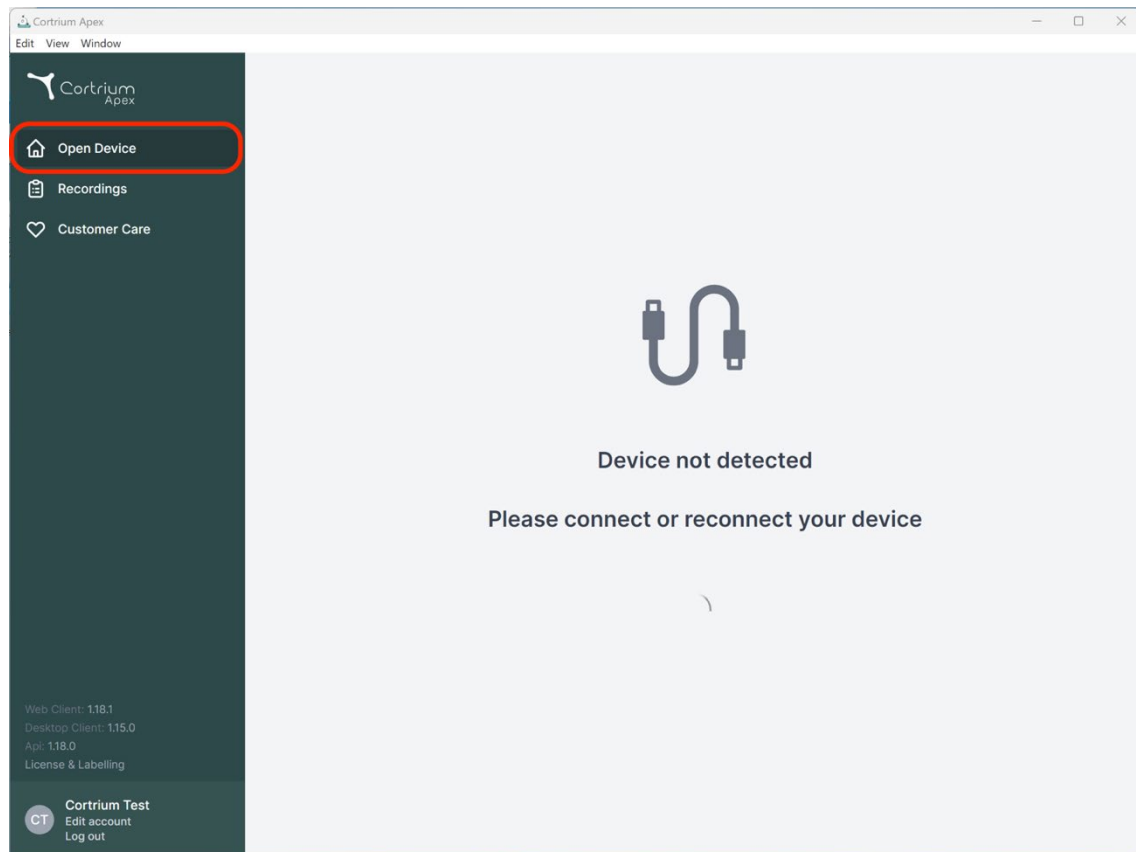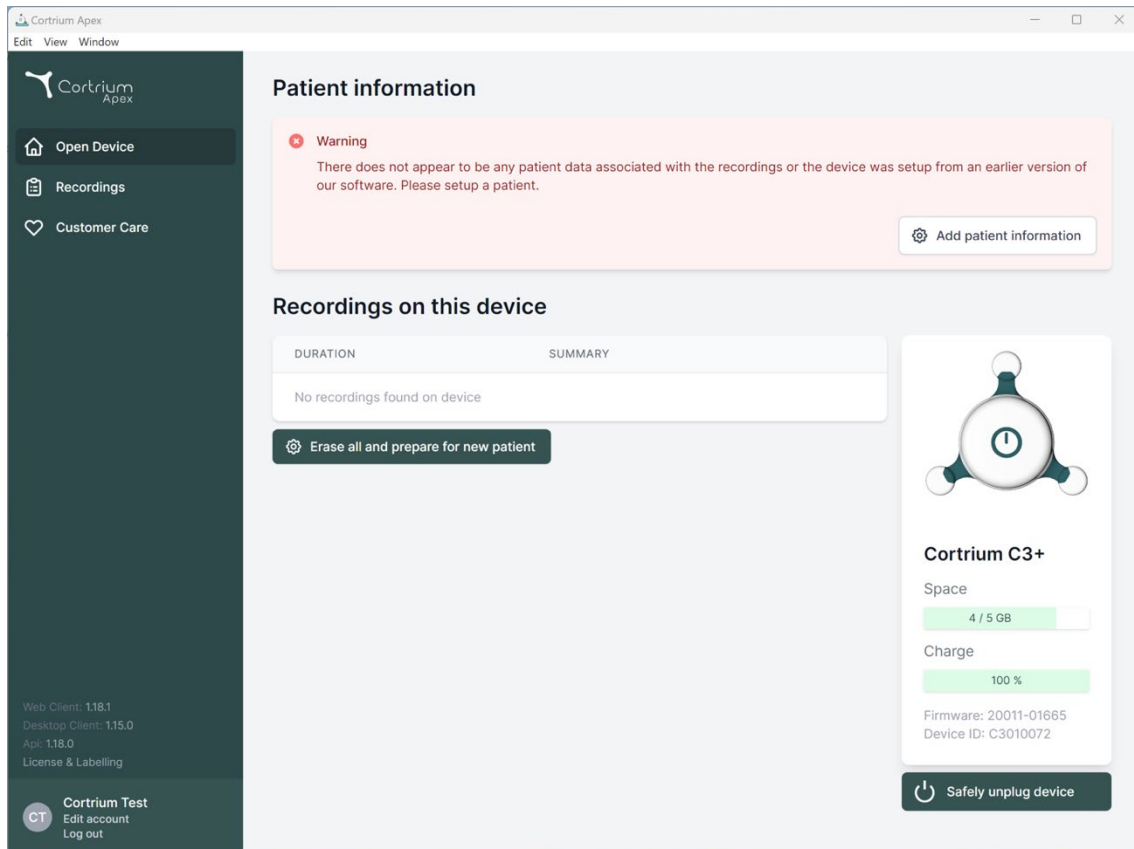
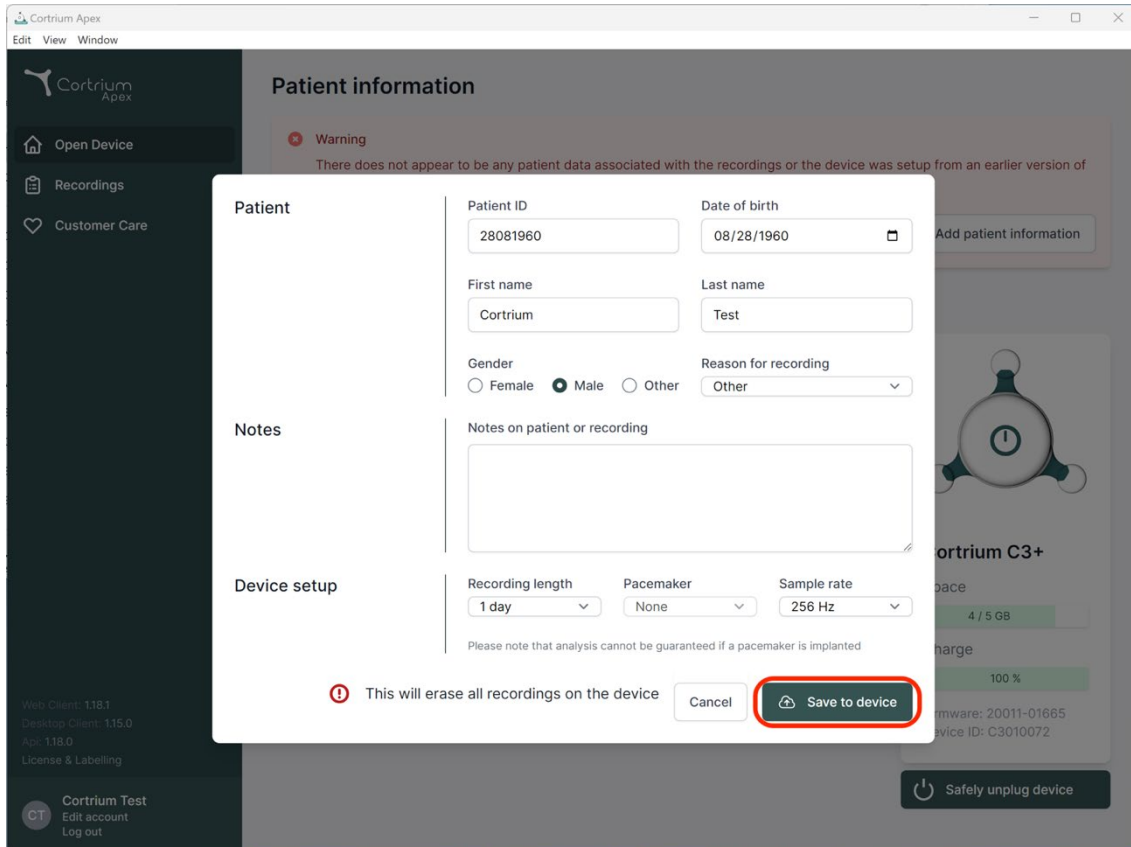
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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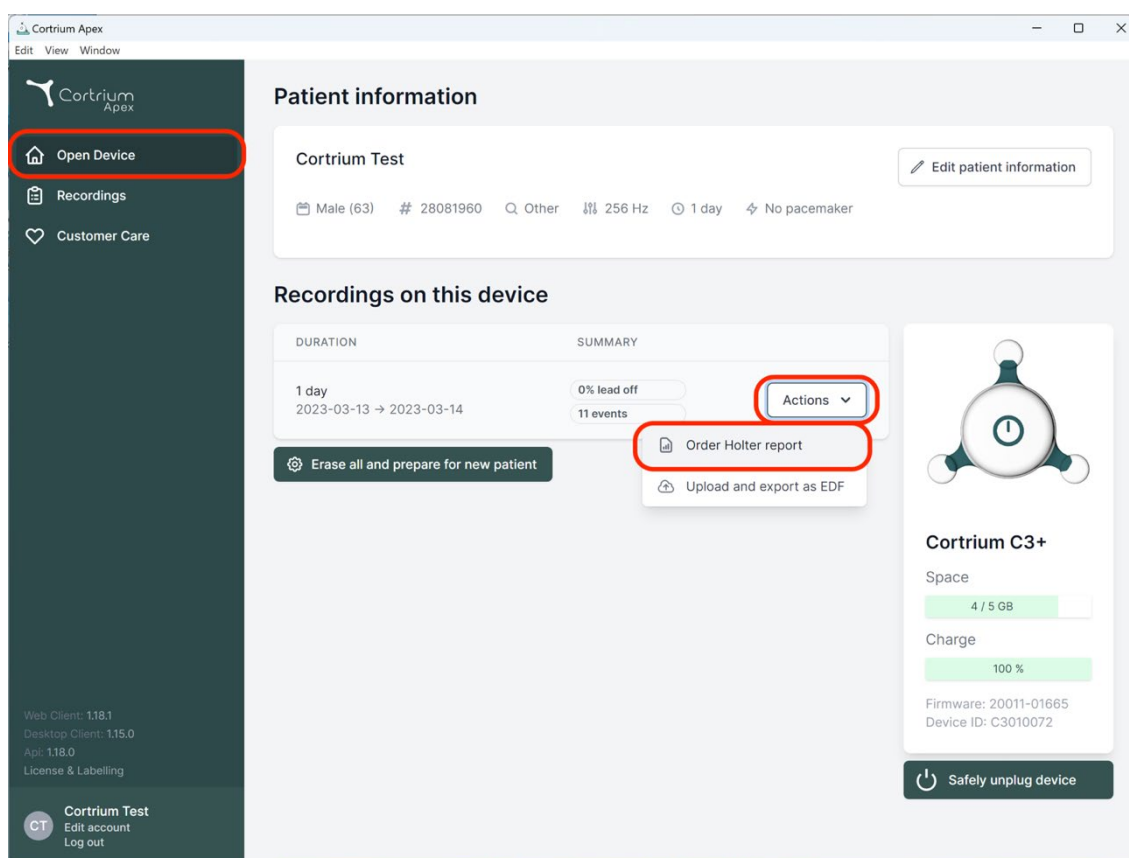
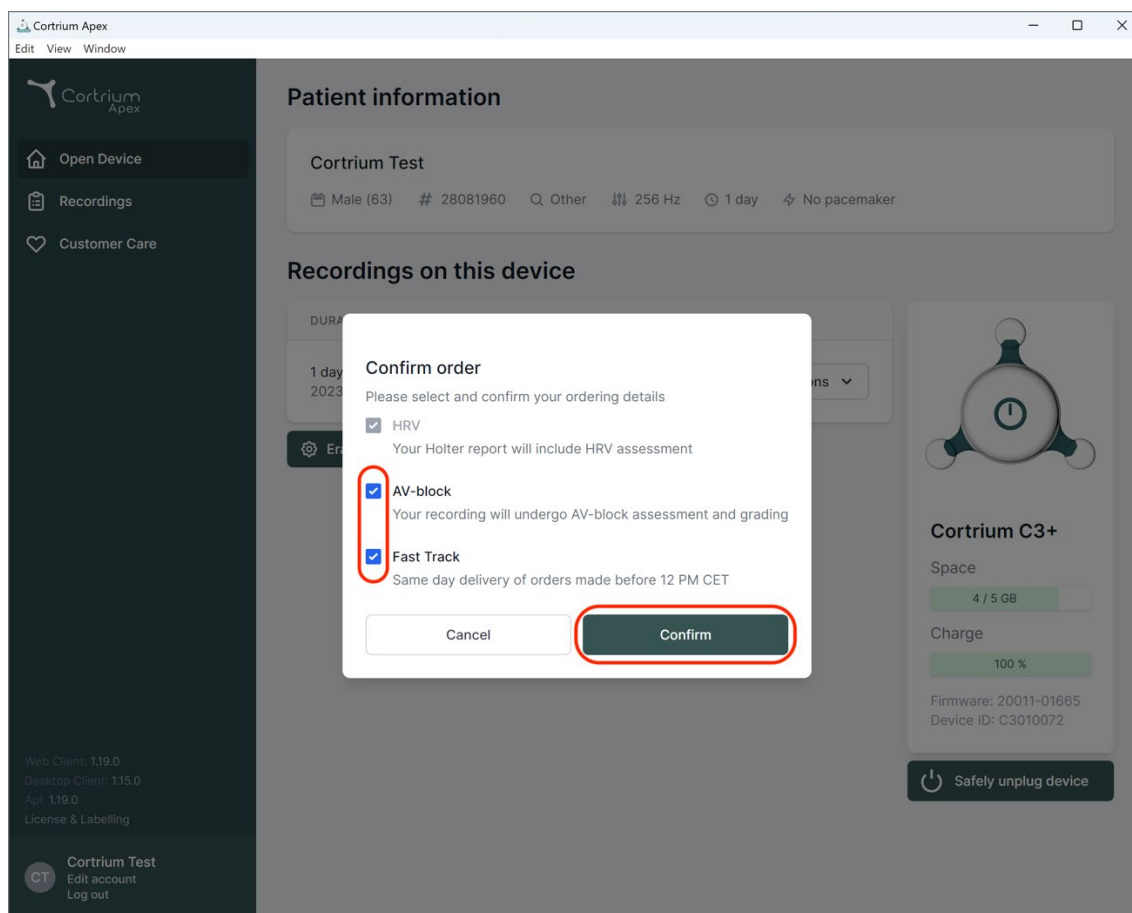
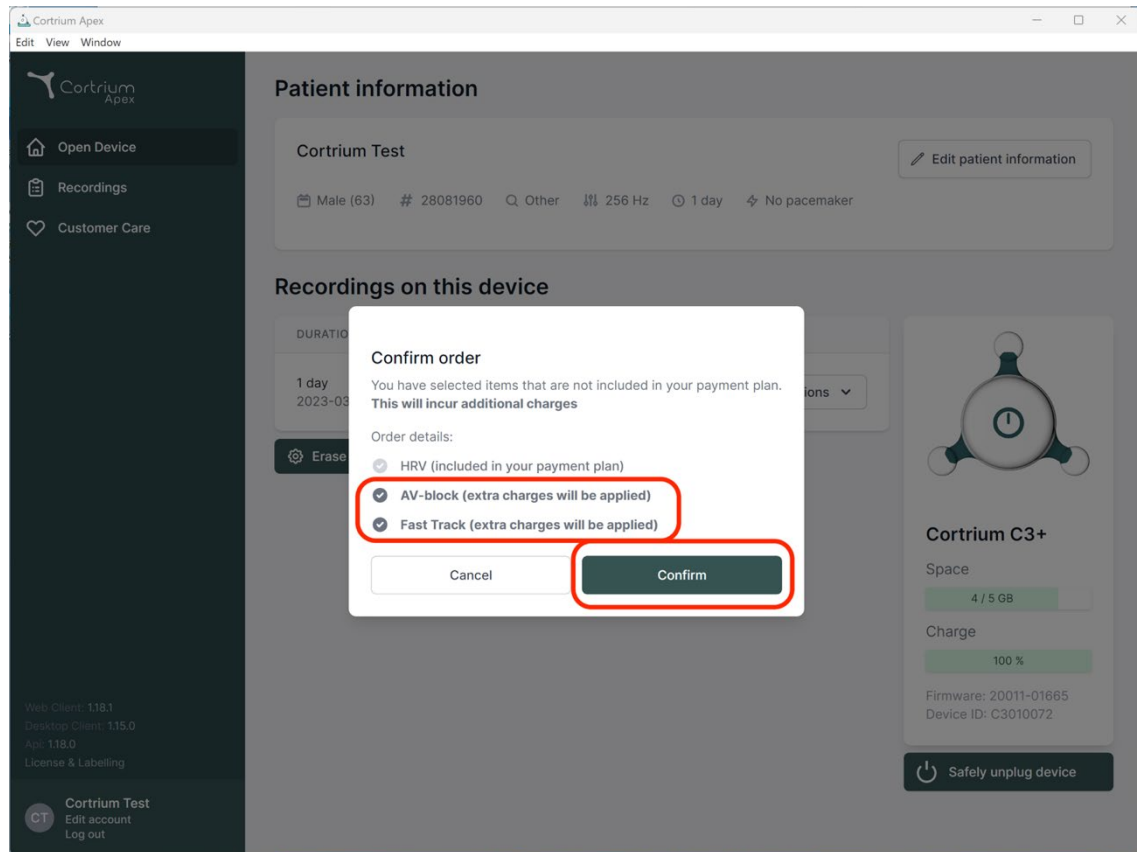
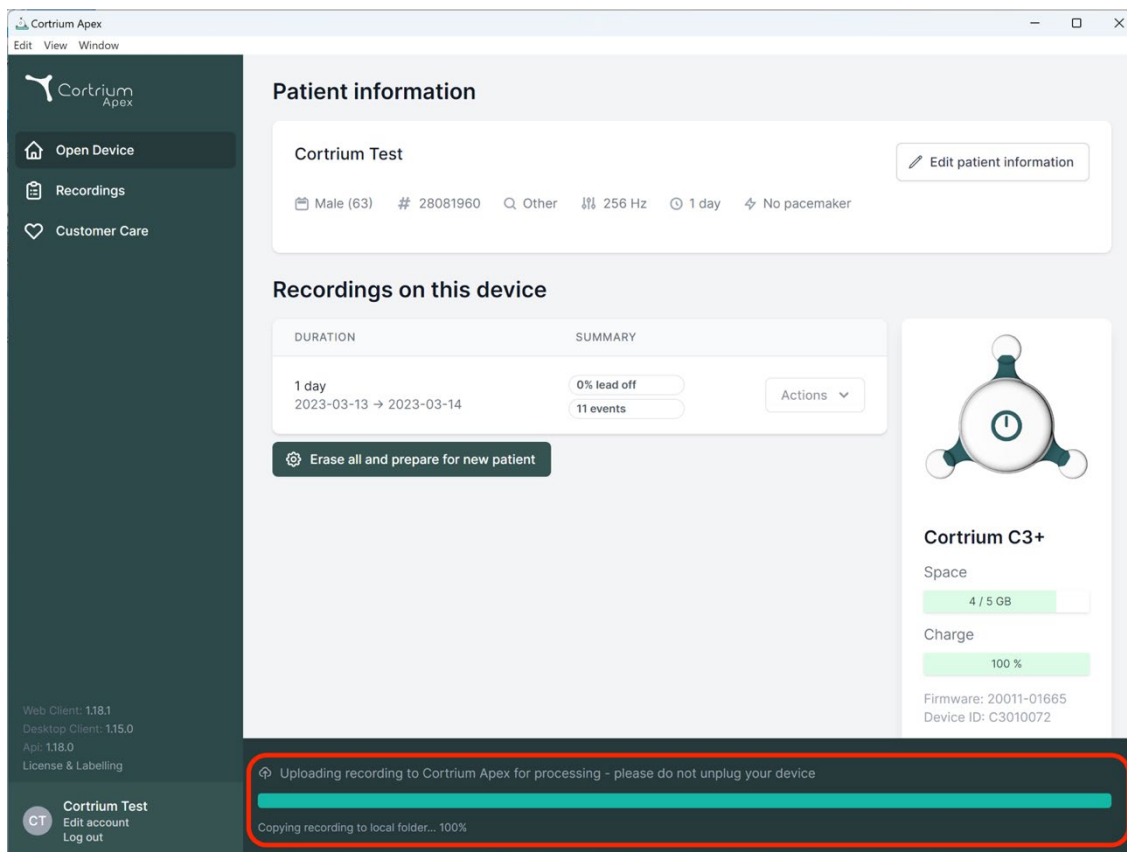
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including patient demographics and test parameters. Below this is a 'Recordings on this device' section with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A red box highlights a status bar at the bottom of the interface, which contains the text: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. On the right side of the interface, there is a device status card for 'Cortrium C3+' showing space (4 / 5 GB) and charge (100 %) levels, along with firmware and device ID information.

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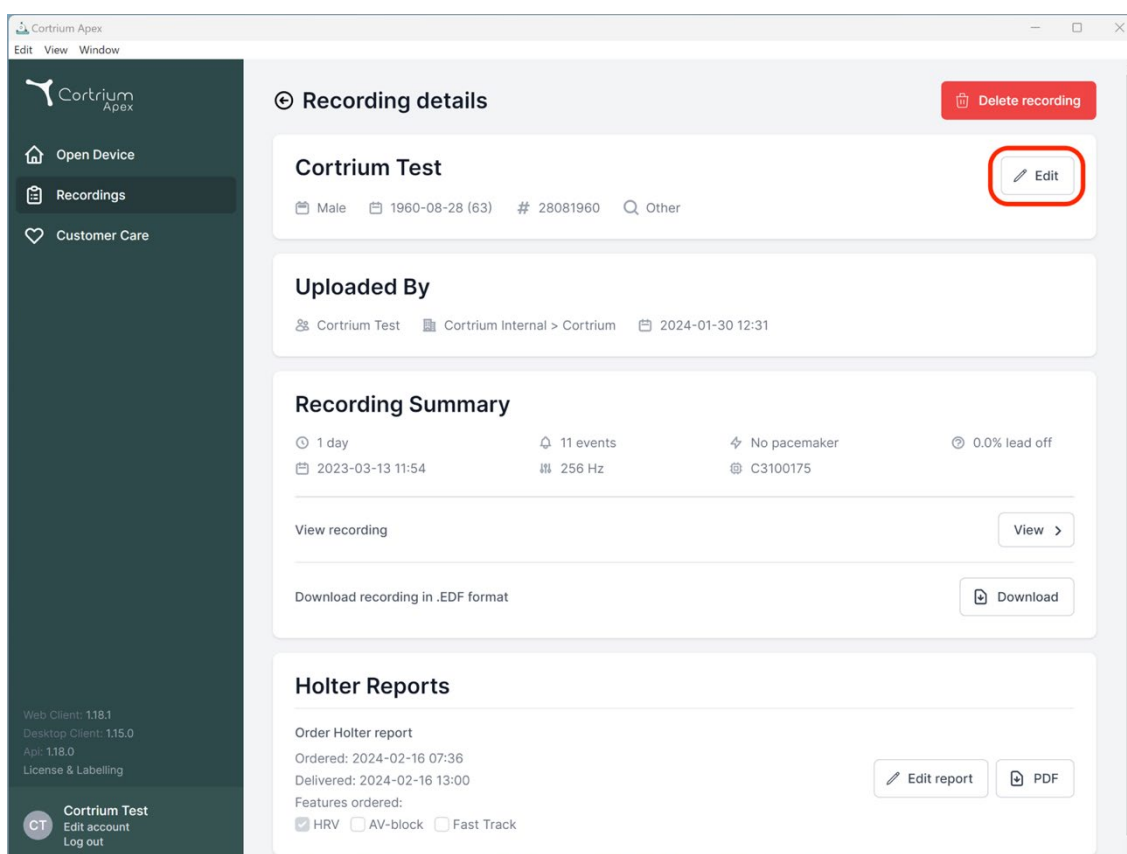
Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary
 1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports
 Order Holter report
 Ordered: 2024-02-16 07:36
 Delivered: 2024-02-16 13:00
 Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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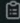
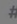
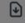
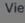
Order Holter report


Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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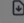
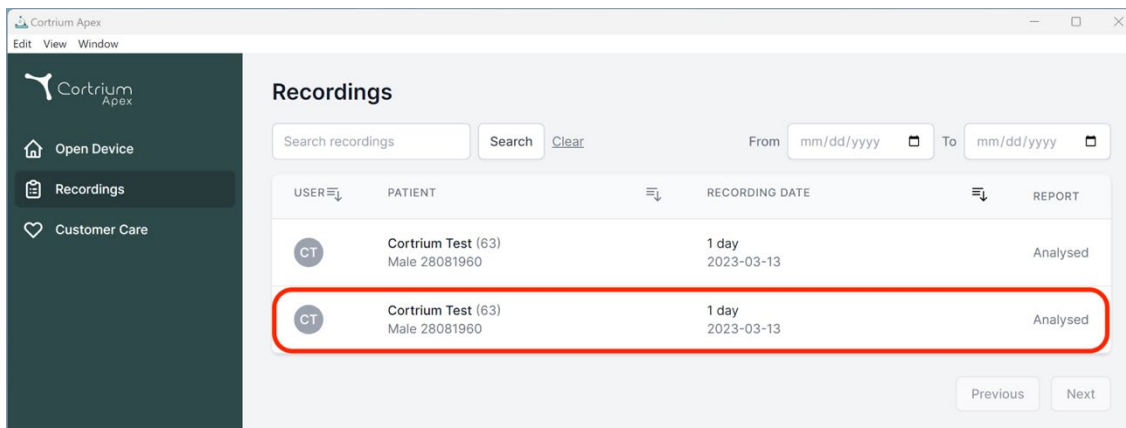
Recordings

Search recordings Search [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

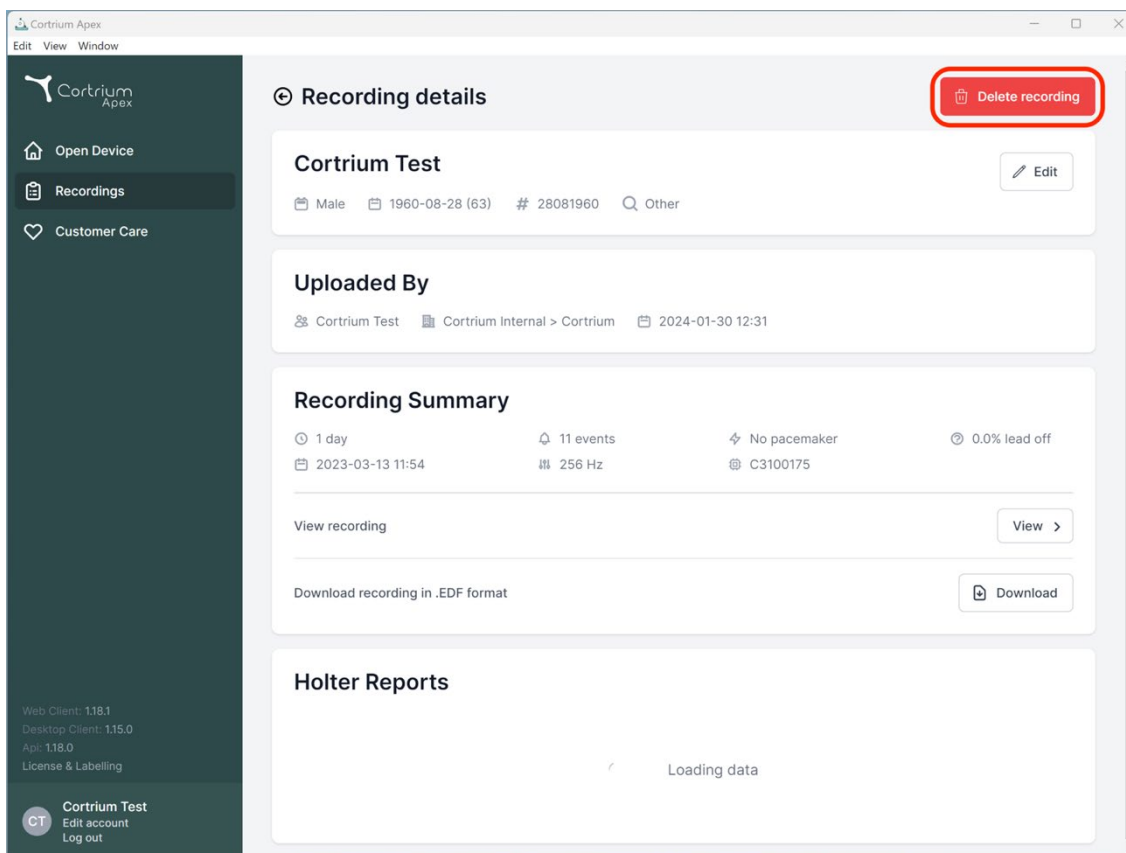Previous Next

Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Loading data

Web Client: 1.18.1
Desktop Client: 1.15.0
Api: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Open Device

Recordings

Customer Care

Web Client: 118.1

Desktop Client: 115.0

App: 118.0

License & Labelling

CT Cortrium Test

Edit account

Log out

Recording details

Delete recording

Cortrium Test

Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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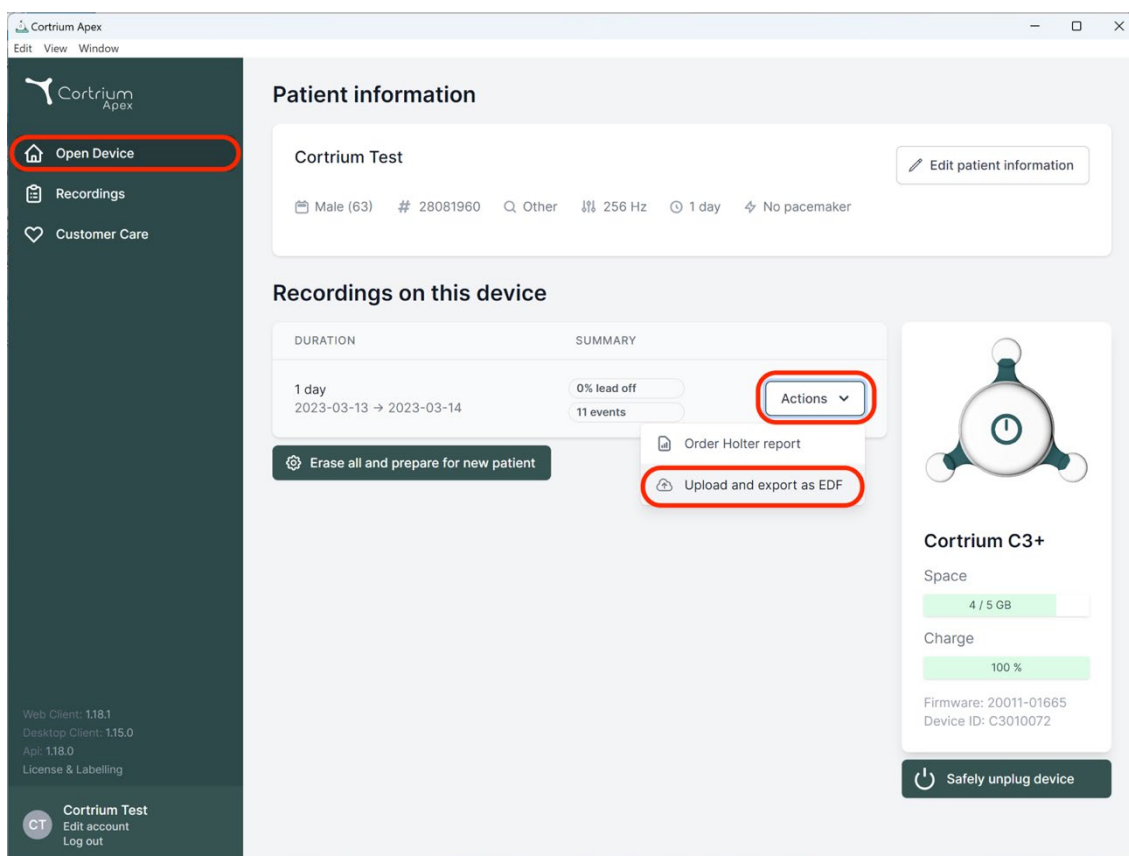
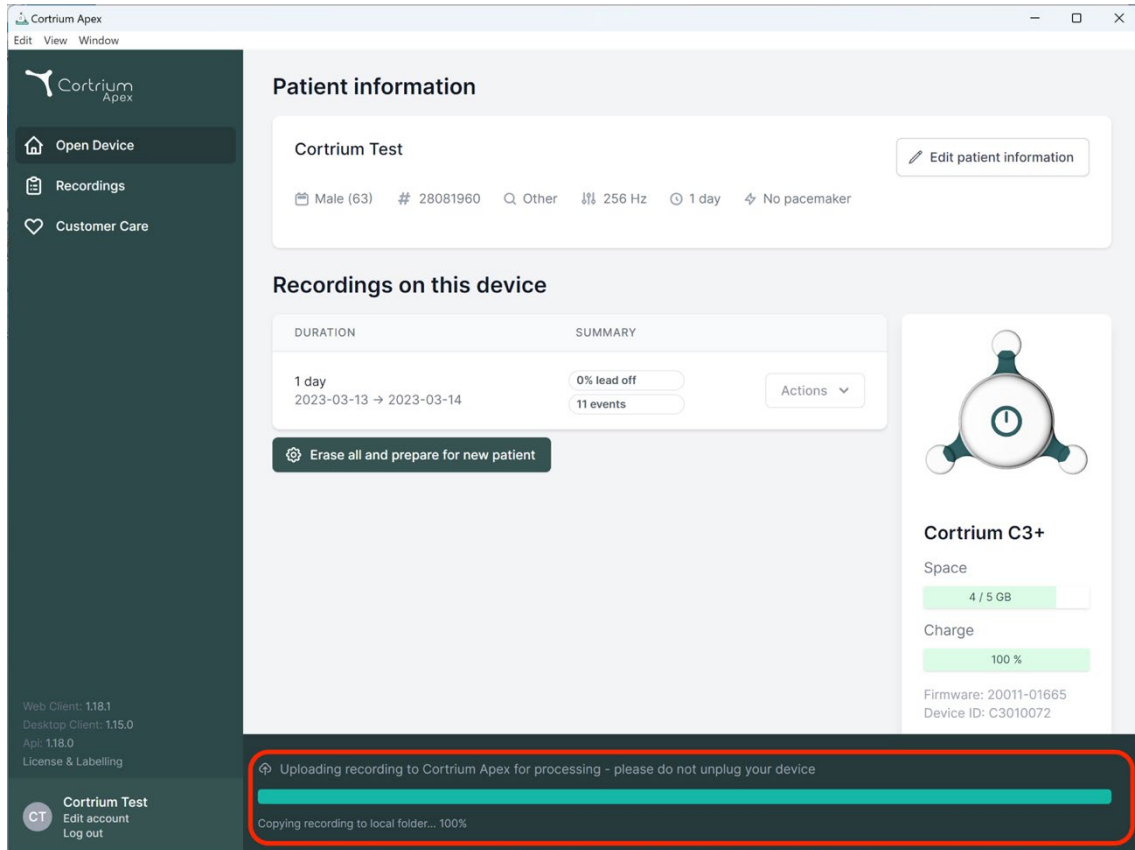
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

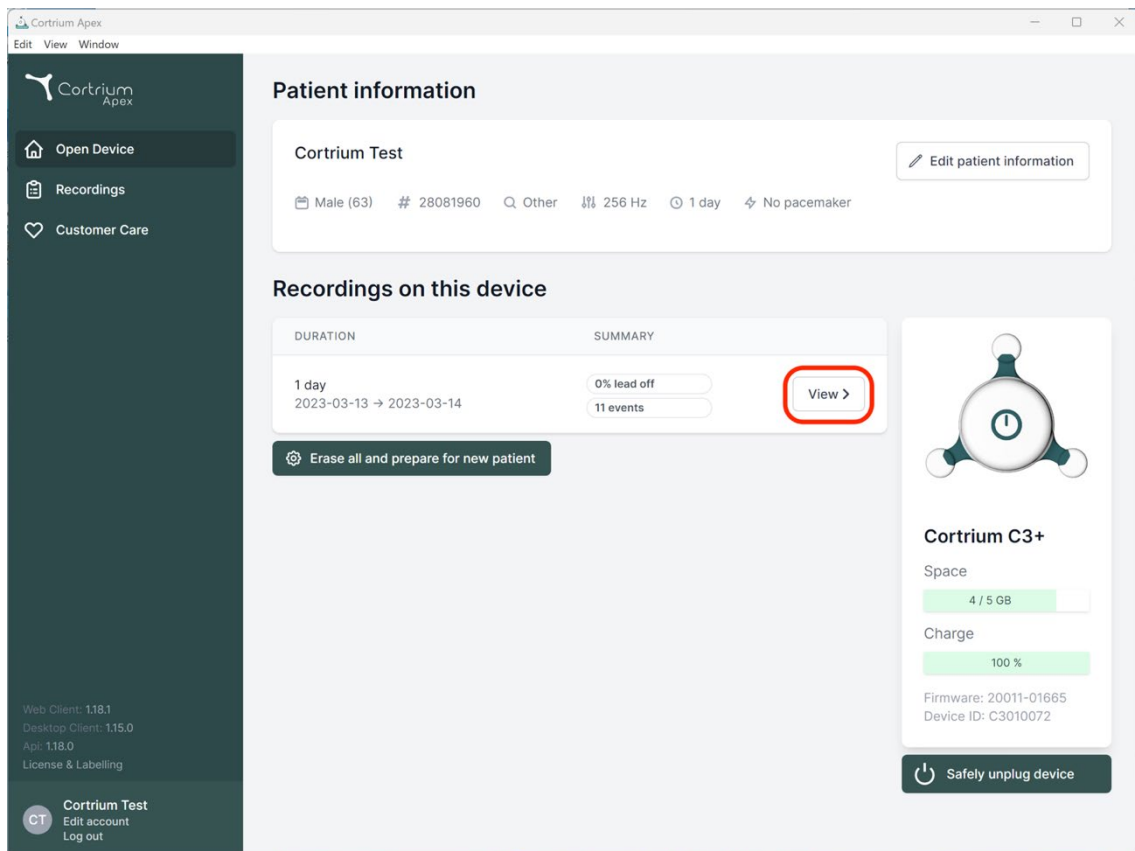
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
Api: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
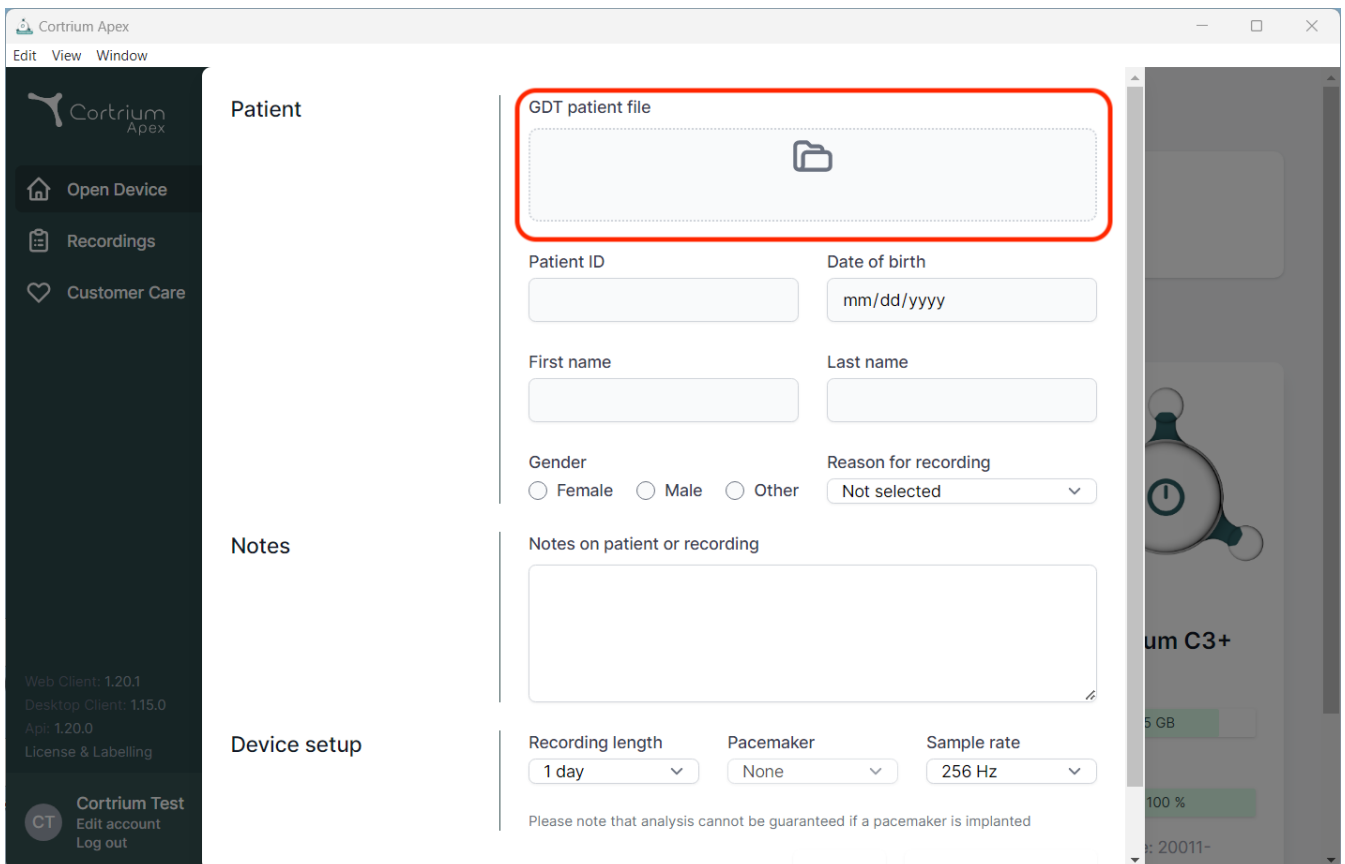
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.20.1, Desktop Client: 1.15.0, App: 1.20.0) and a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area is divided into three sections: 'Patient', 'Notes', and 'Device setup'. The 'Patient' section contains a 'GDT patient file' upload area (highlighted with a red box), followed by input fields for 'Patient ID', 'Date of birth' (with a 'mm/dd/yyyy' placeholder), 'First name', and 'Last name'. Below these are radio buttons for 'Gender' (Female, Male, Other) and a dropdown for 'Reason for recording' (currently 'Not selected'). The 'Notes' section has a text area for 'Notes on patient or recording'. The 'Device setup' section includes dropdowns for 'Recording length' (set to '1 day'), 'Pacemaker' (set to 'None'), and 'Sample rate' (set to '256 Hz'). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. On the right edge, a partial view of a medical device screen shows 'um C3+' and a '100 %' indicator.

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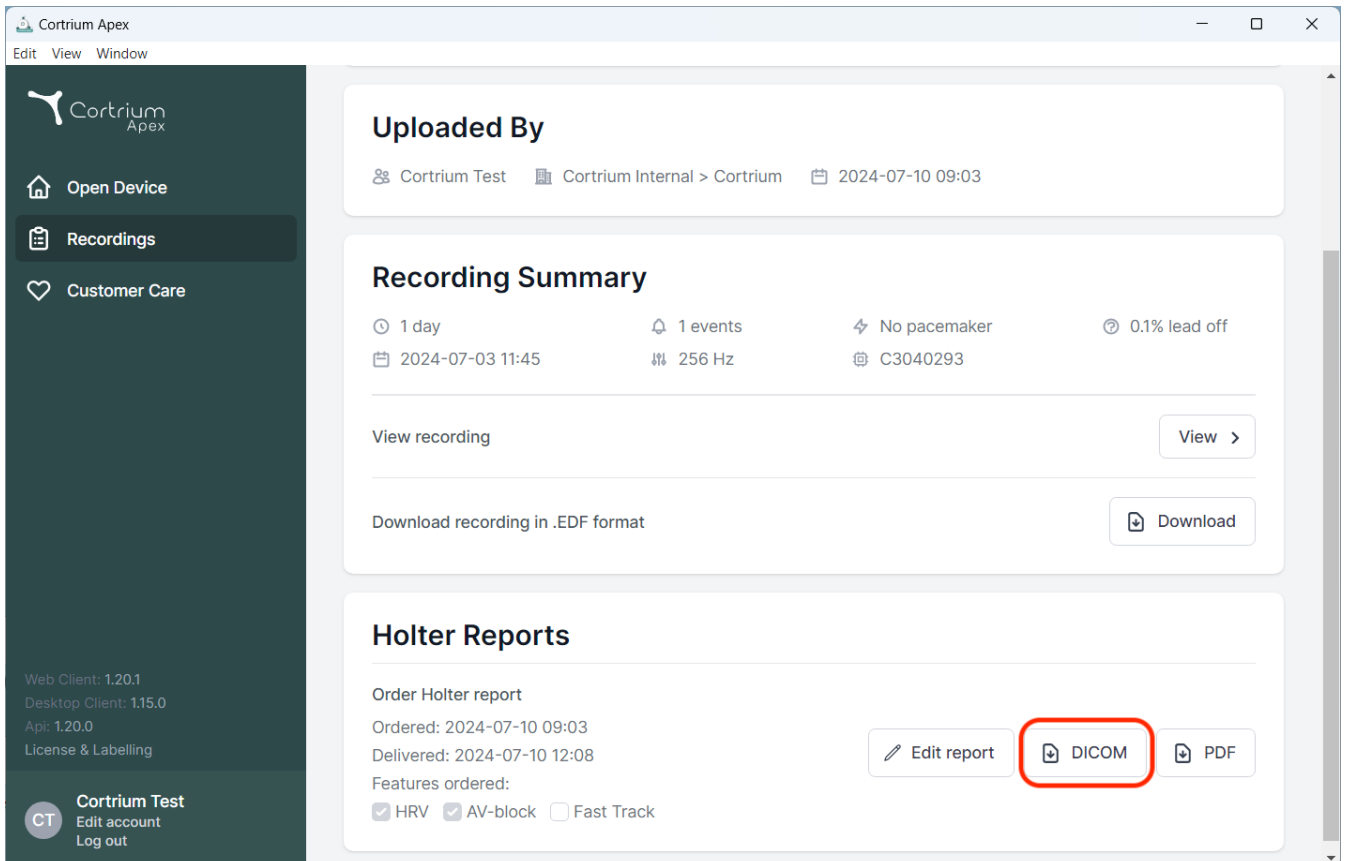The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By', 'Recording Summary', and 'Holter Reports'.

Uploaded By
Cortrium Test | Cortrium Internal > Cortrium | 2024-07-10 09:03

Recording Summary
🕒 1 day | 🔔 1 events | ⚡ No pacemaker | ⚙️ 0.1% lead off
📅 2024-07-03 11:45 | 📡 256 Hz | 🏷️ C3040293

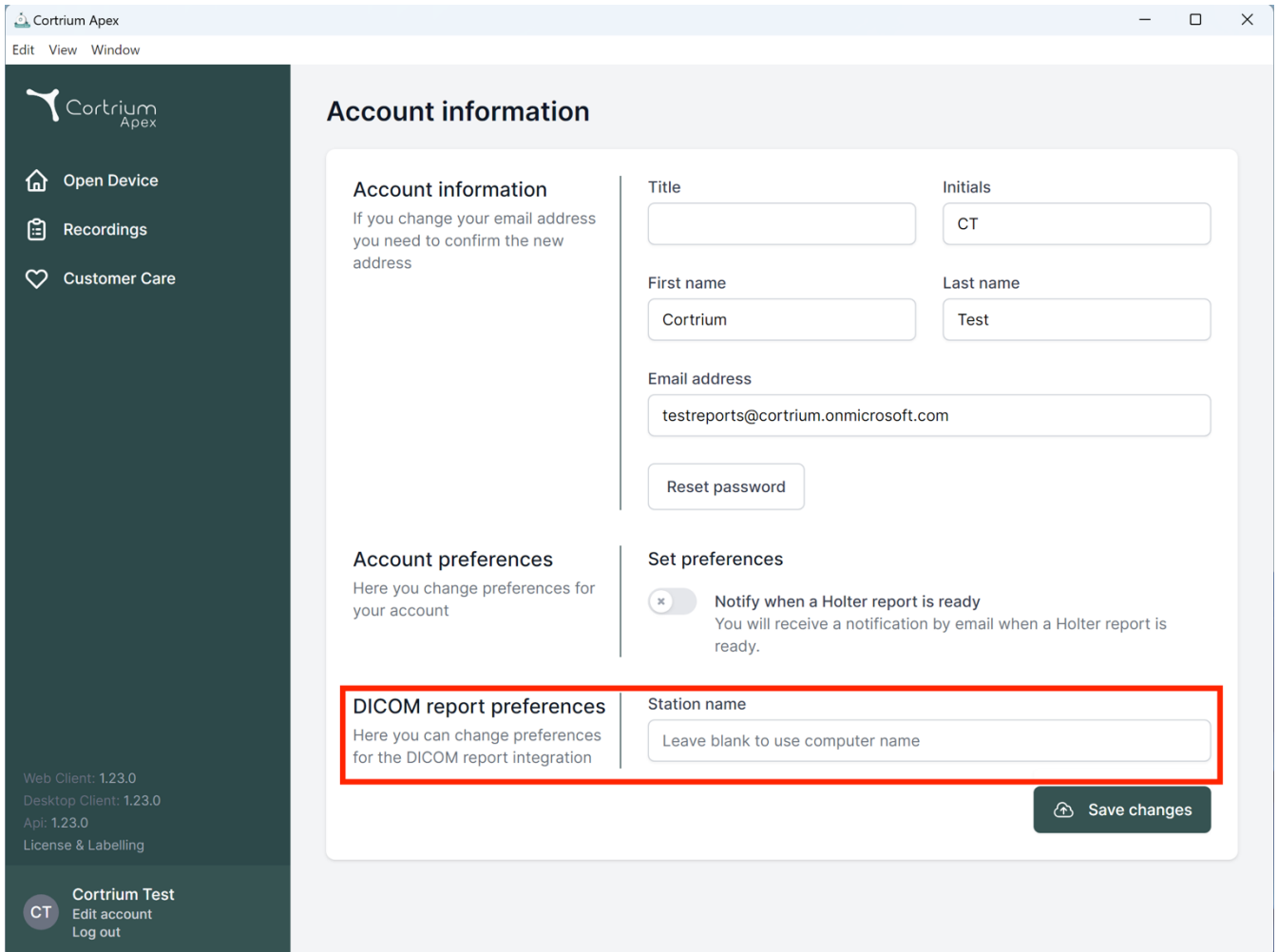
View recording [View >]
Download recording in .EDF format [Download]

Holter Reports
Order Holter report
Ordered: 2024-07-10 09:03
Delivered: 2024-07-10 12:08
Features ordered:
☒ HRV ☒ AV-block ☐ Fast Track

Buttons: [Edit report] [DICOM] [PDF]. The 'DICOM' button is highlighted with a red rectangle.

Footer (left sidebar):
Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling
CT Cortrium Test
Edit account
Log out

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Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

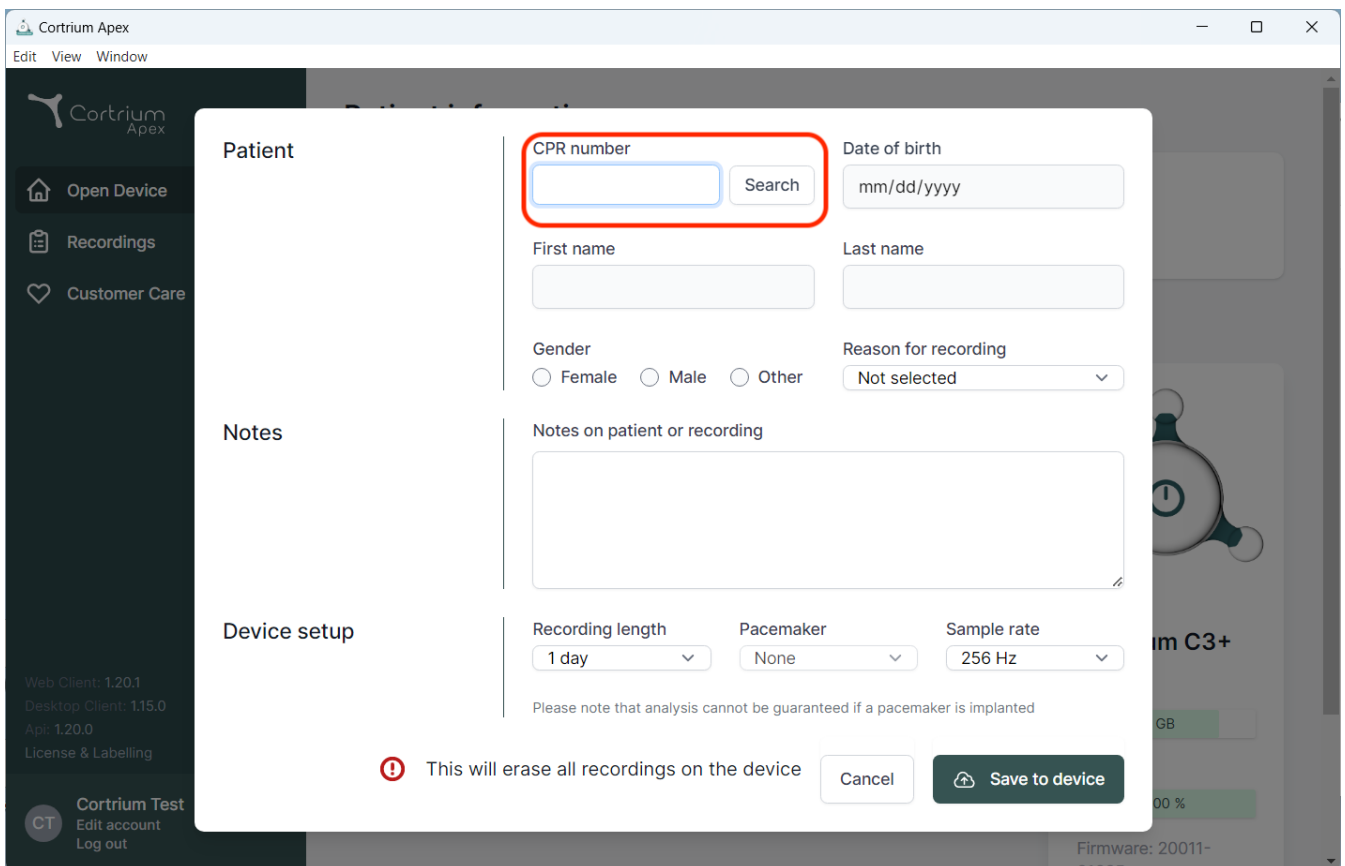
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient search and data entry. The "CPR number" field is highlighted with a red box, and a "Search" button is next to it. The form includes fields for "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), "Notes on patient or recording", "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A warning message at the bottom states: "This will erase all recordings on the device". The background shows the Cortrium Apex sidebar with options like "Open Device", "Recordings", and "Customer Care".

Patient

CPR number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

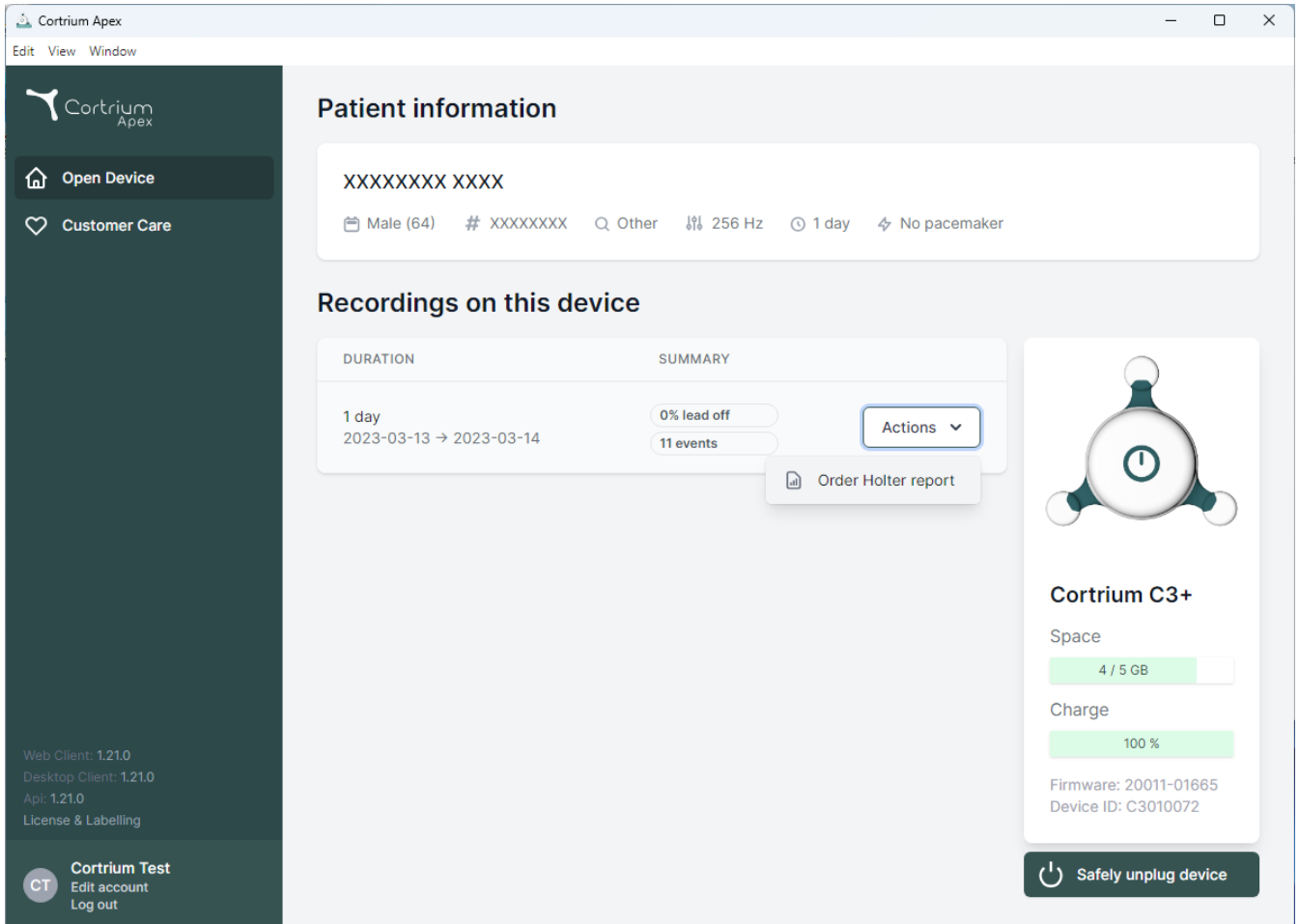
Recording length Pacemaker Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device" and "Customer Care". The main content area is titled "Patient information" and shows a patient ID "XXXXXXXX XXXX". Below this, patient details are listed: "Male (64)", "# XXXXXXXX", "Other", "256 Hz", "1 day", and "No pacemaker". The "Recordings on this device" section features a table with columns "DURATION" and "SUMMARY". The table shows a recording for "1 day" from "2023-03-13" to "2023-03-14", with "0% lead off" and "11 events". An "Actions" dropdown menu is visible, with "Order Holter report" selected. On the right, a device status card for "Cortrium C3+" shows "Space" at "4 / 5 GB" and "Charge" at "100 %". It also displays "Firmware: 20011-01665" and "Device ID: C3010072". A "Safely unplug device" button is at the bottom right. The bottom left of the sidebar shows version information: "Web Client: 1.21.0", "Desktop Client: 1.21.0", "Api: 1.21.0", and "License & Labelling". A "Cortrium Test" section includes "Edit account" and "Log out" links.

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

Actions

Order Holter report

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

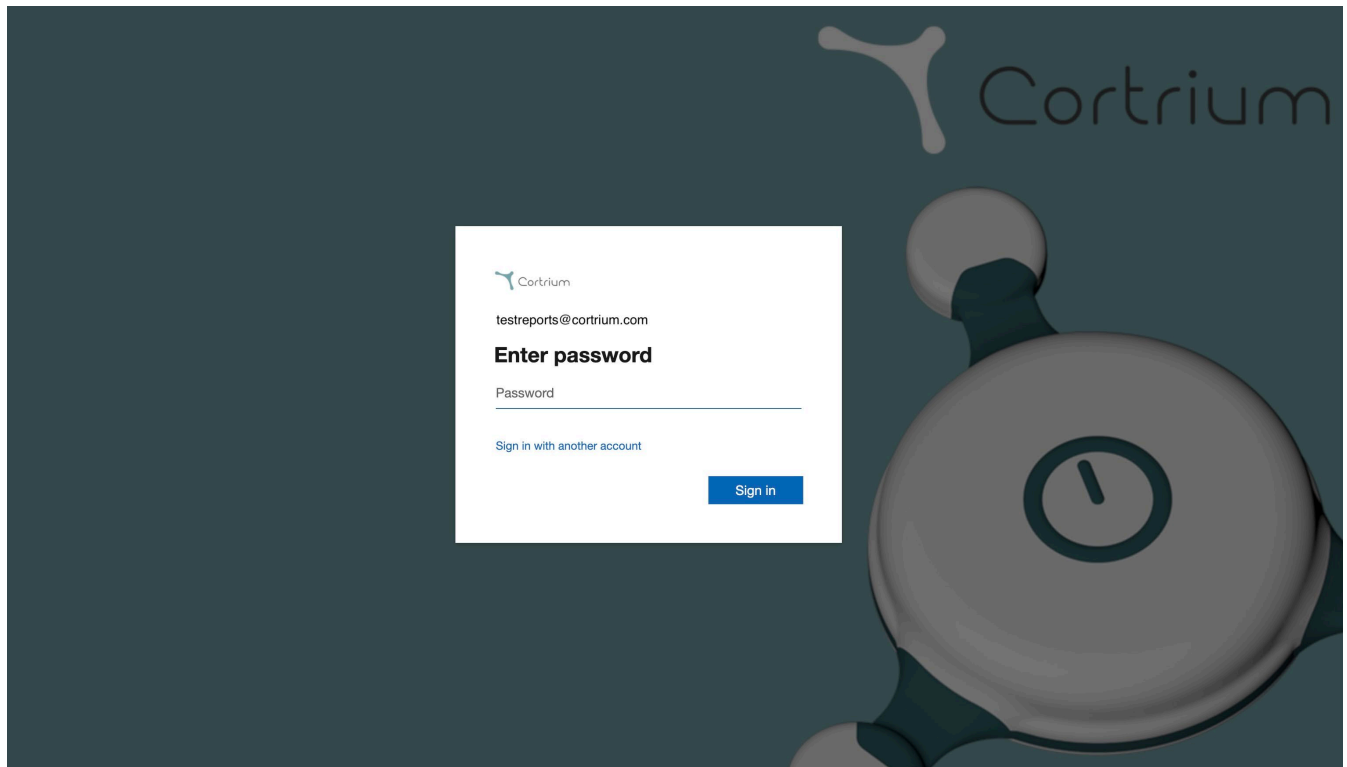
Safely unplug device

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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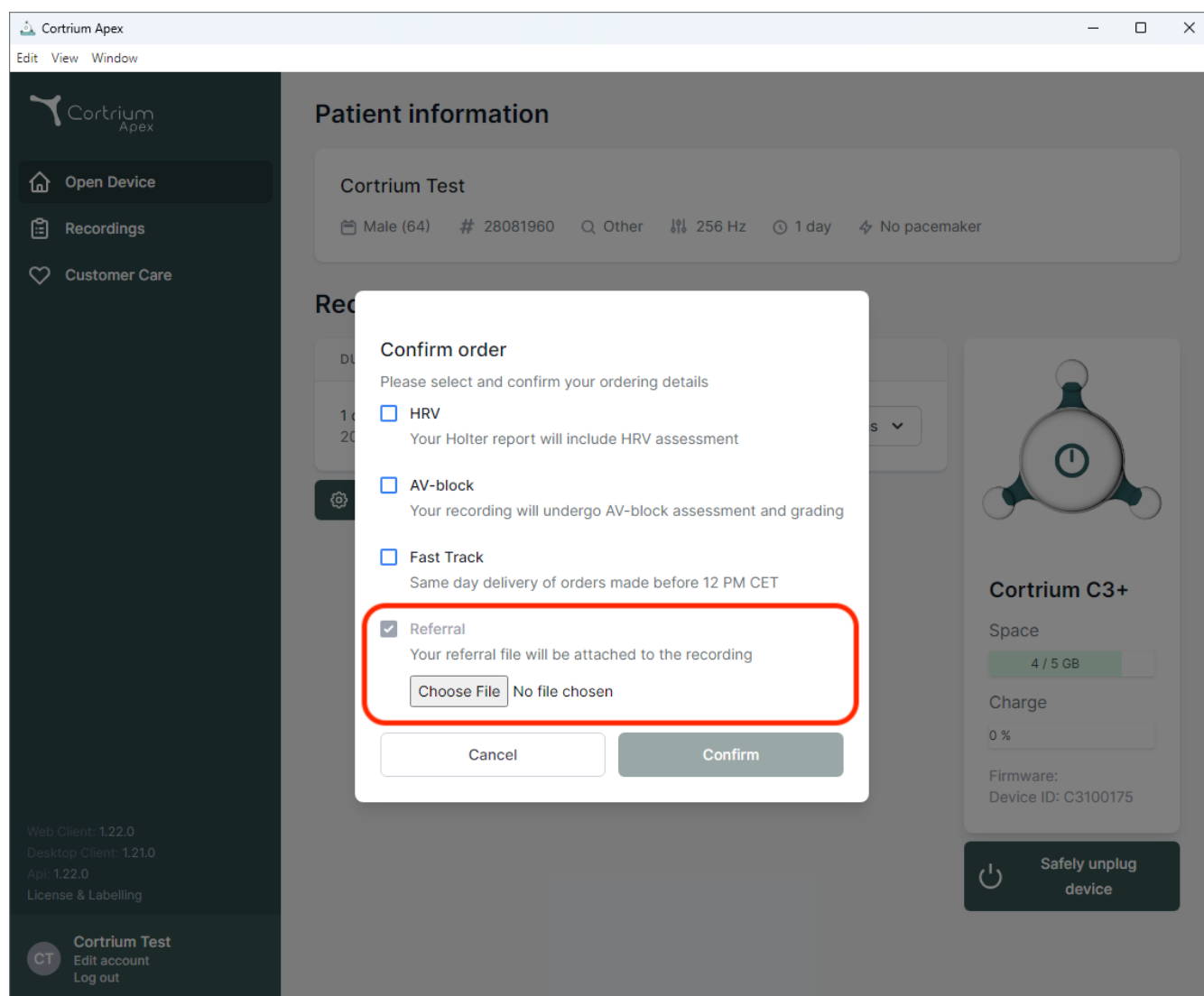
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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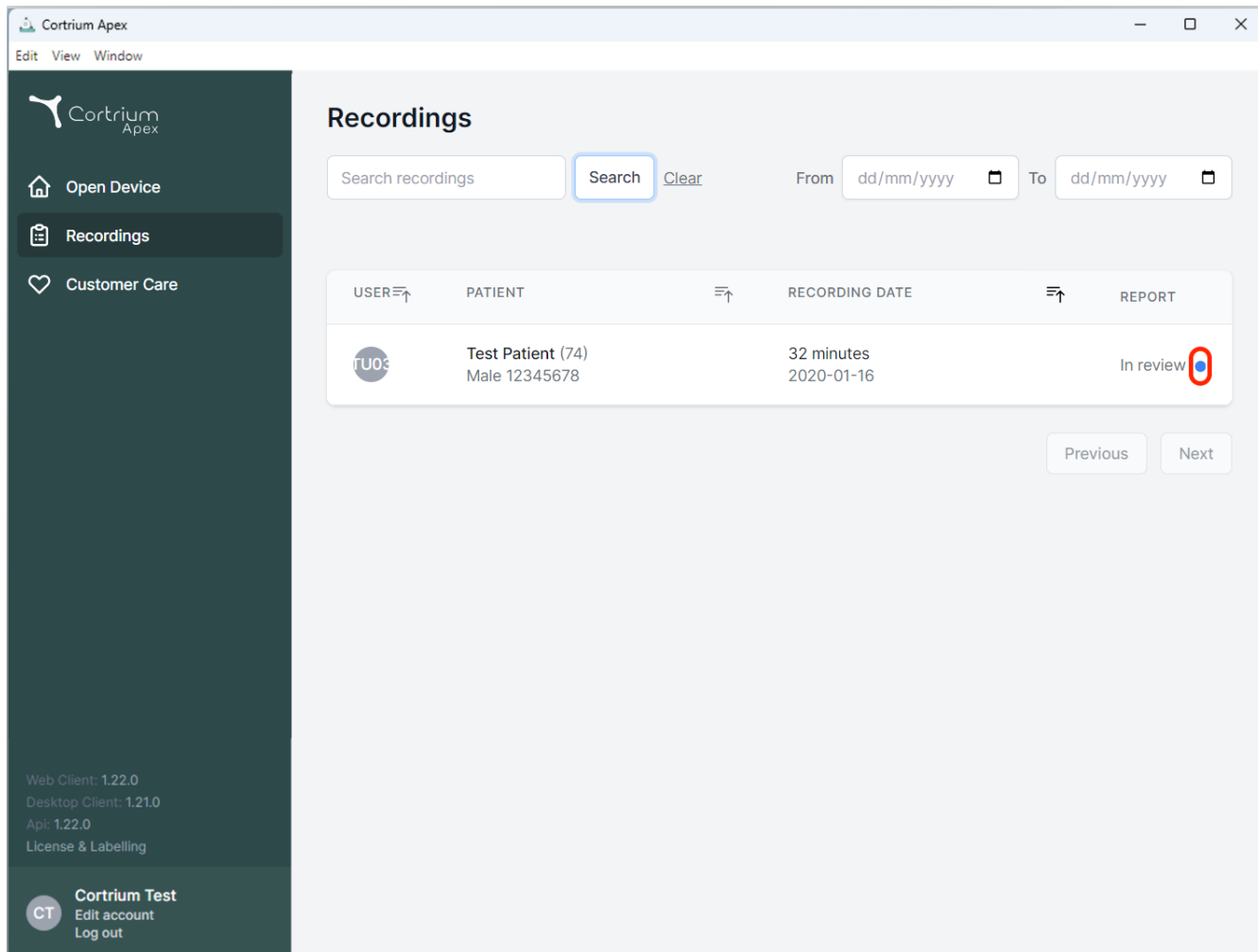
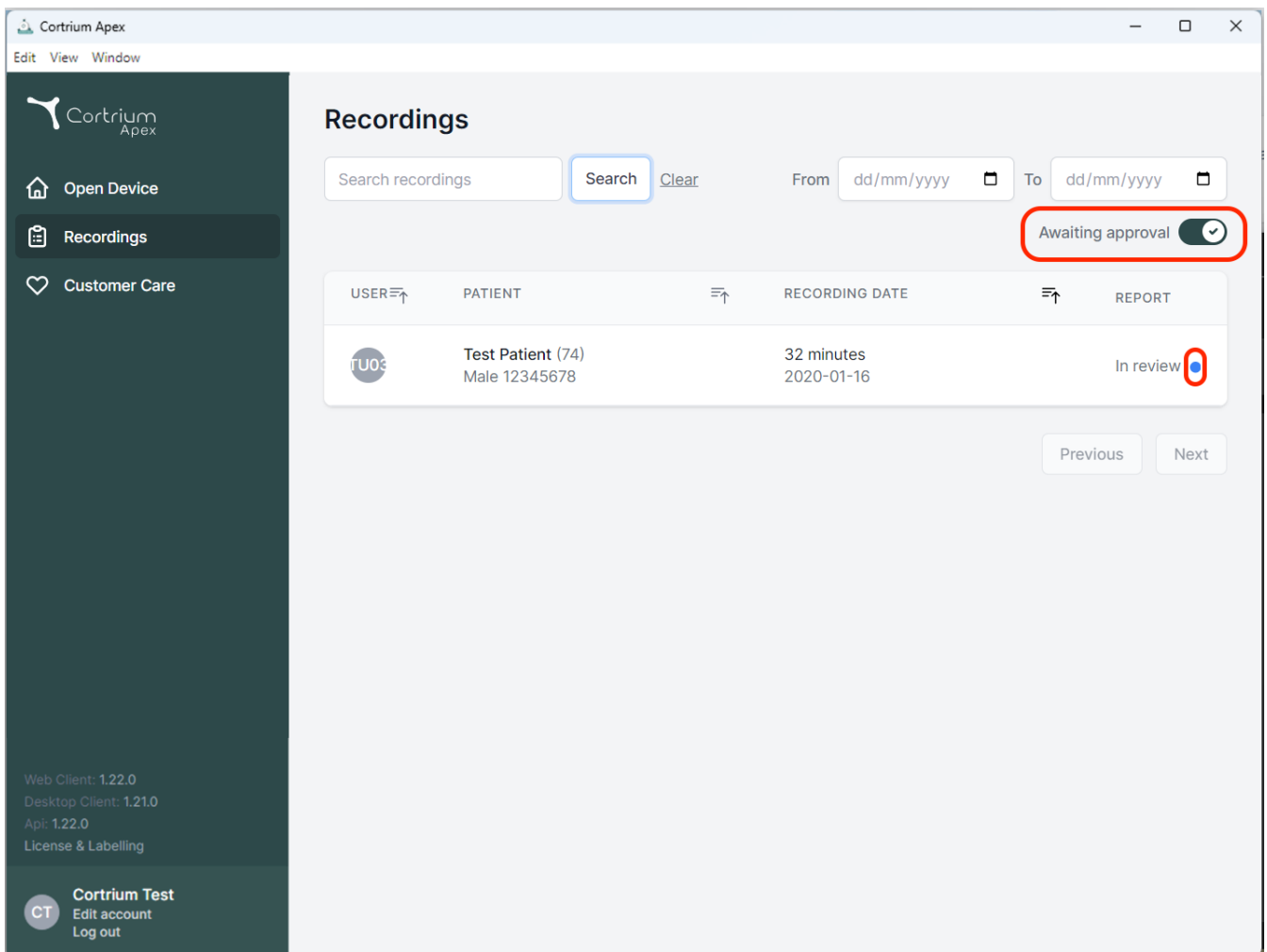
Recordings

Search recordings **Search** [Clear](#) From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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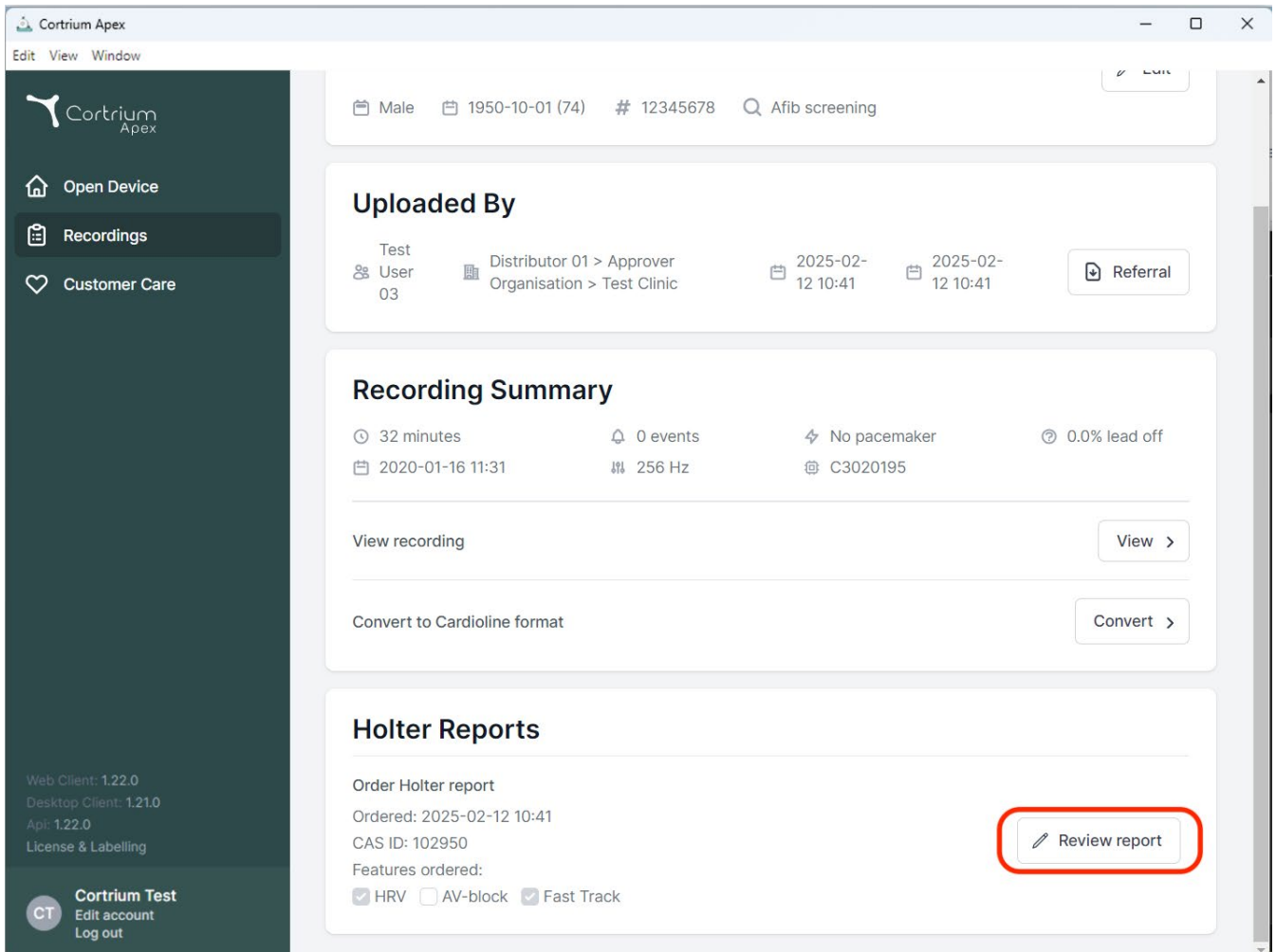The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes the Cortrium Apex logo and a menu with 'Edit', 'View', and 'Window'. The left sidebar contains navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section shows 'Test User 03' as the distributor, with dates 2025-02-12 10:41 and a 'Referral' button. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a date of 2020-01-16 11:31. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows an 'Order Holter report' with an order date of 2025-02-12 10:41, CAS ID 102950, and features ordered: HRV, AV-block, and Fast Track. A red box highlights the 'Review report' button.

Cortrium Apex

Edit View Window

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

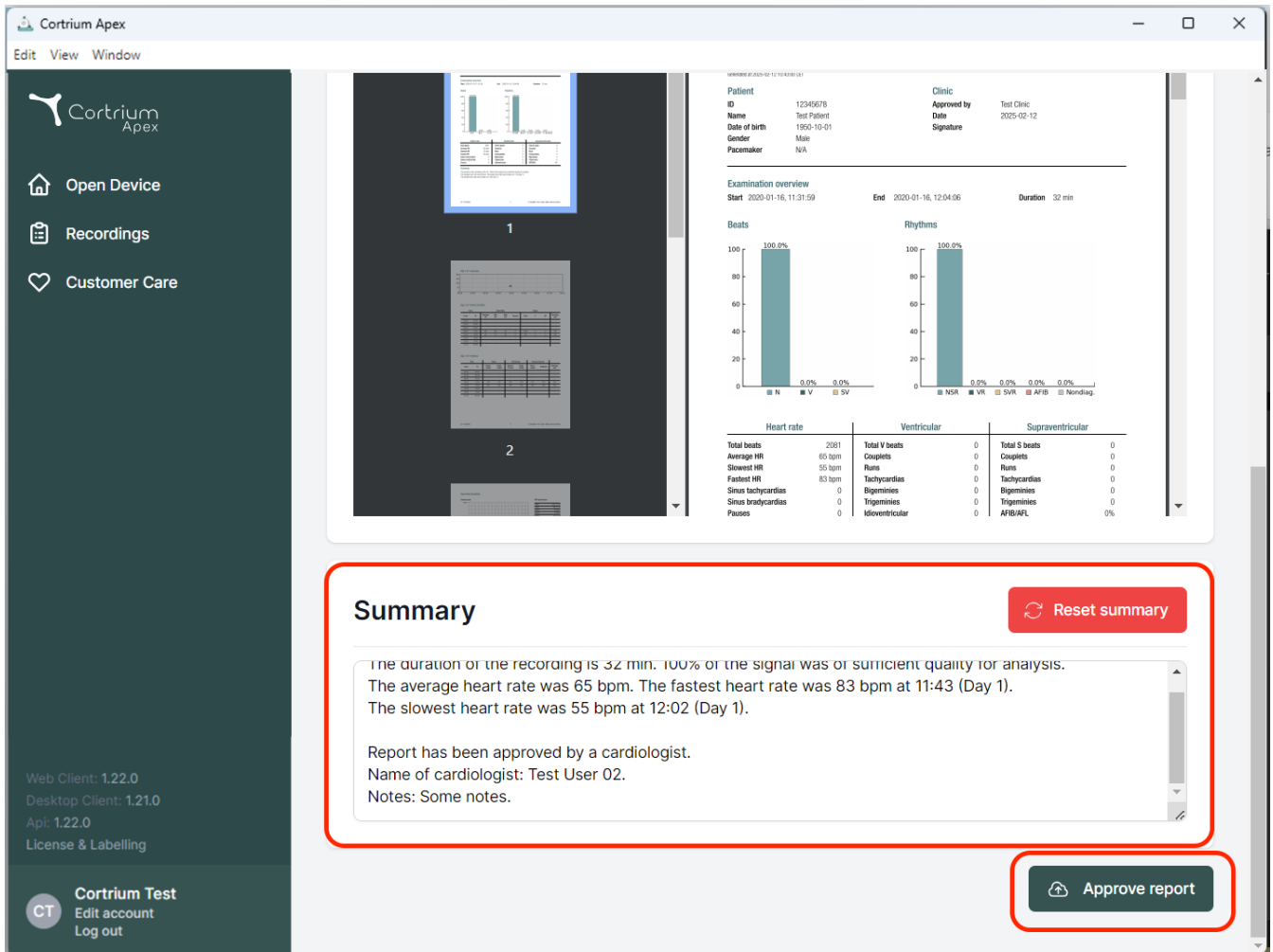
Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Patient
ID: 12345678
Name: Test Patient
Date of birth: 1950-10-01
Gender: Male
Pacemaker: N/A

Clinic
Approved by: Test Clinic
Date: 2025-02-12
Signature:

Examination overview
Start: 2020-01-16, 11:31:59
End: 2020-01-16, 12:04:06
Duration: 32 min

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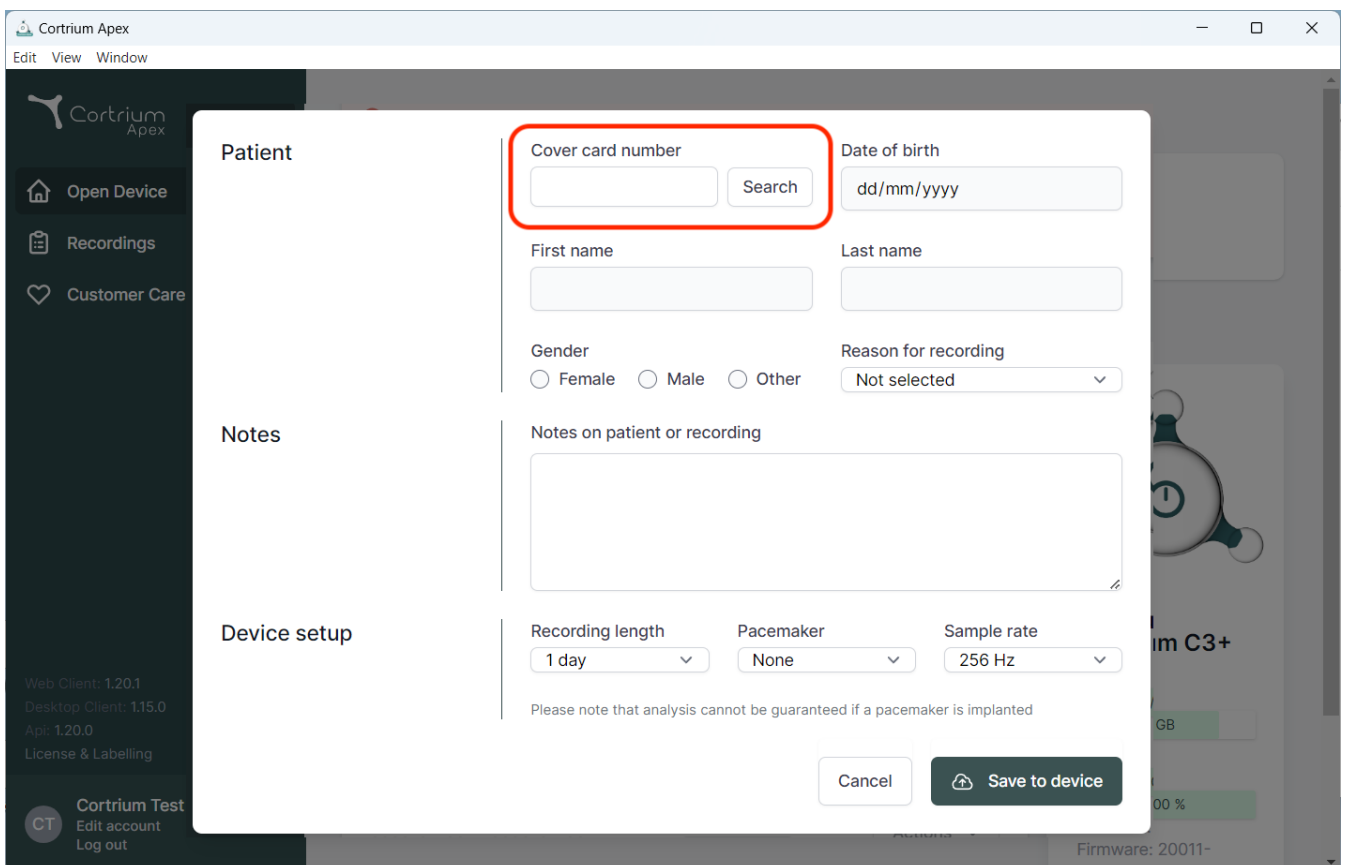
The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, allowing for patient information entry. The modal is divided into three main sections: "Patient", "Notes", and "Device setup".

Patient Section:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup options, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

The background interface shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom left corner displays version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right corner shows "Cortrium Test" with options to "Edit account" and "Log out".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark
Email: info@cortrium.com • Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47