Cortrium Apex 1.27

„Cortrium Apex 1.27“

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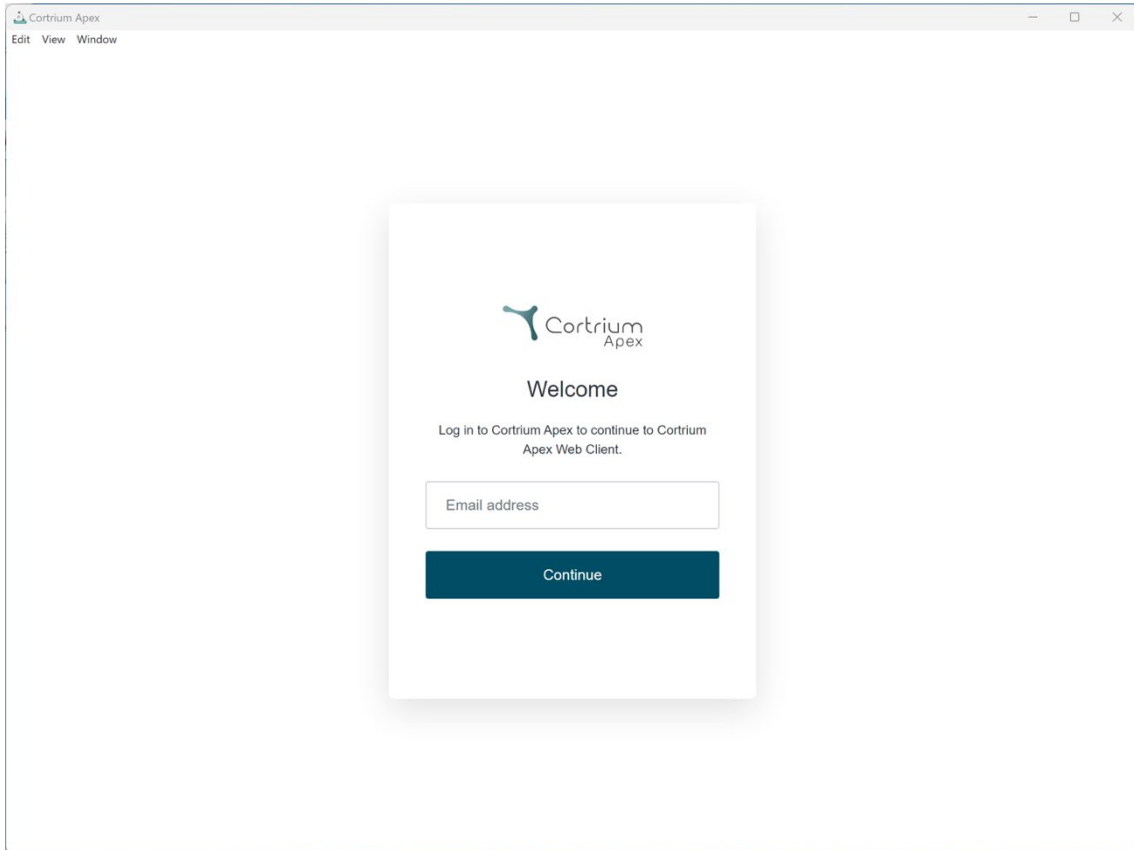
 Cortrium
Apex

Welcome

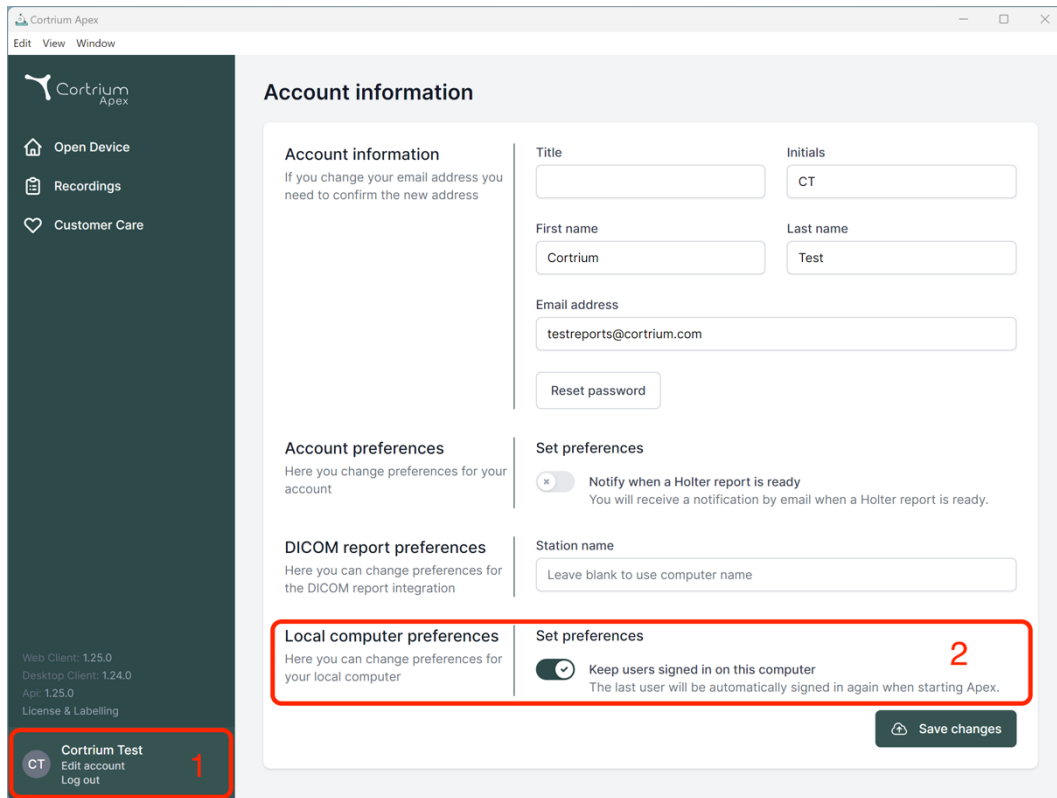
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: Open Device, Recordings, and Customer Care. Below these are version numbers (Web Client: 1.25.0, Desktop Client: 1.24.0, Api: 1.25.0) and a 'License & Labelling' link. At the bottom of the sidebar is a user profile section for 'Cortrium Test' with options to 'Edit account' and 'Log out', marked with a red '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' for changing account settings; 'DICOM report preferences' for integration settings; and 'Local computer preferences' for local device settings. The 'Local computer preferences' section includes a 'Set preferences' area with a toggle for 'Keep users signed in on this computer', which is currently turned on and marked with a red '2'. A 'Save changes' button is located at the bottom right of the preferences section.

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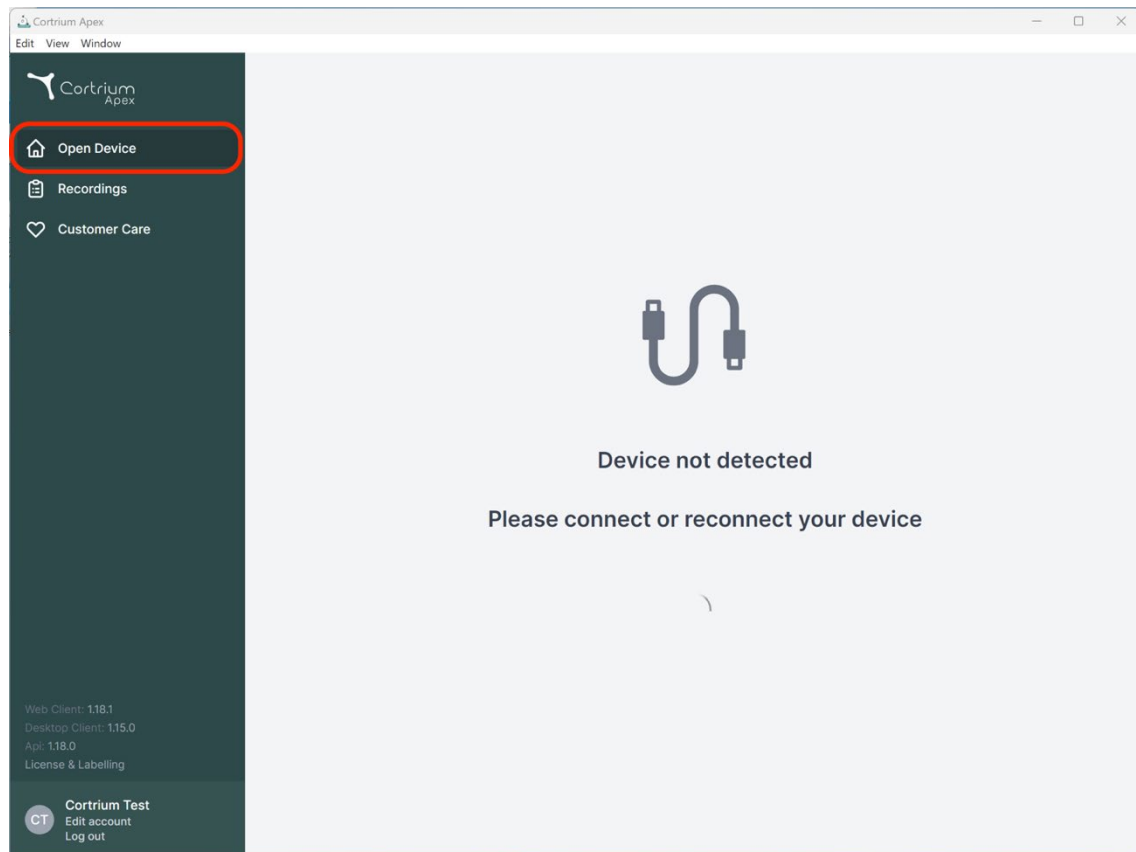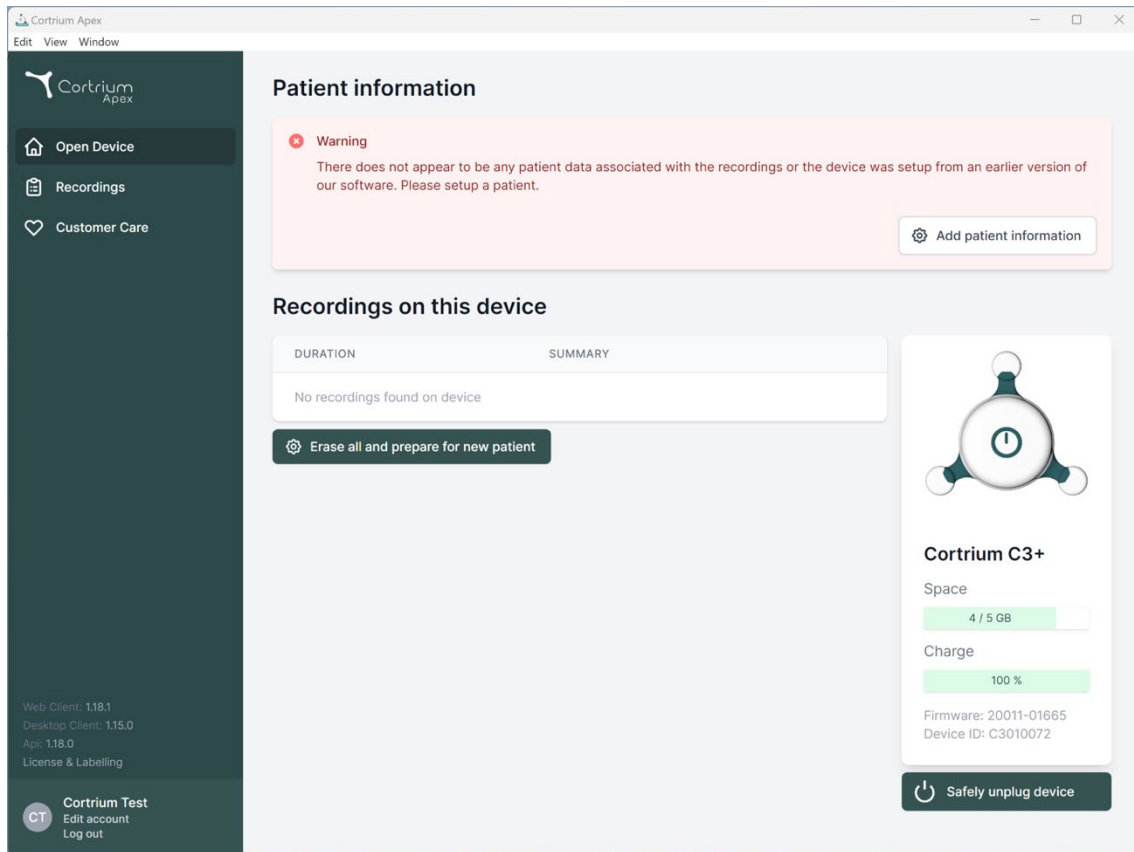
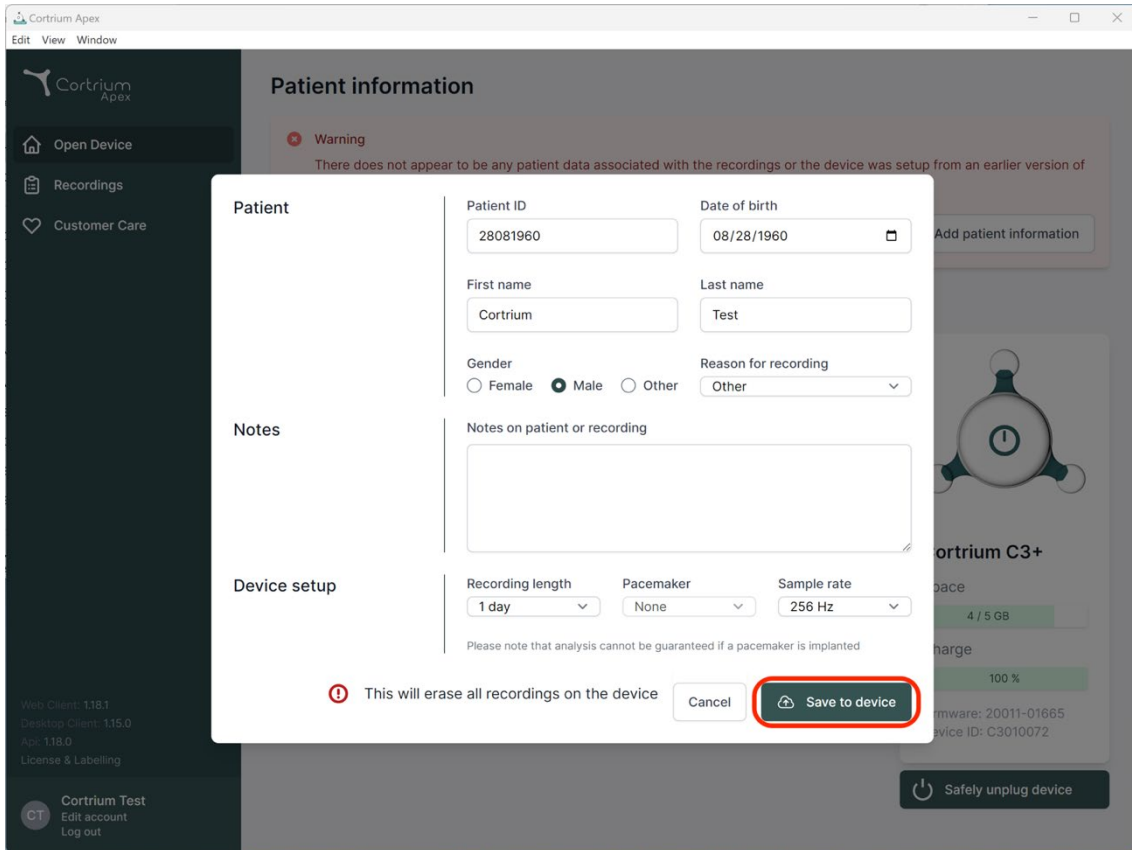
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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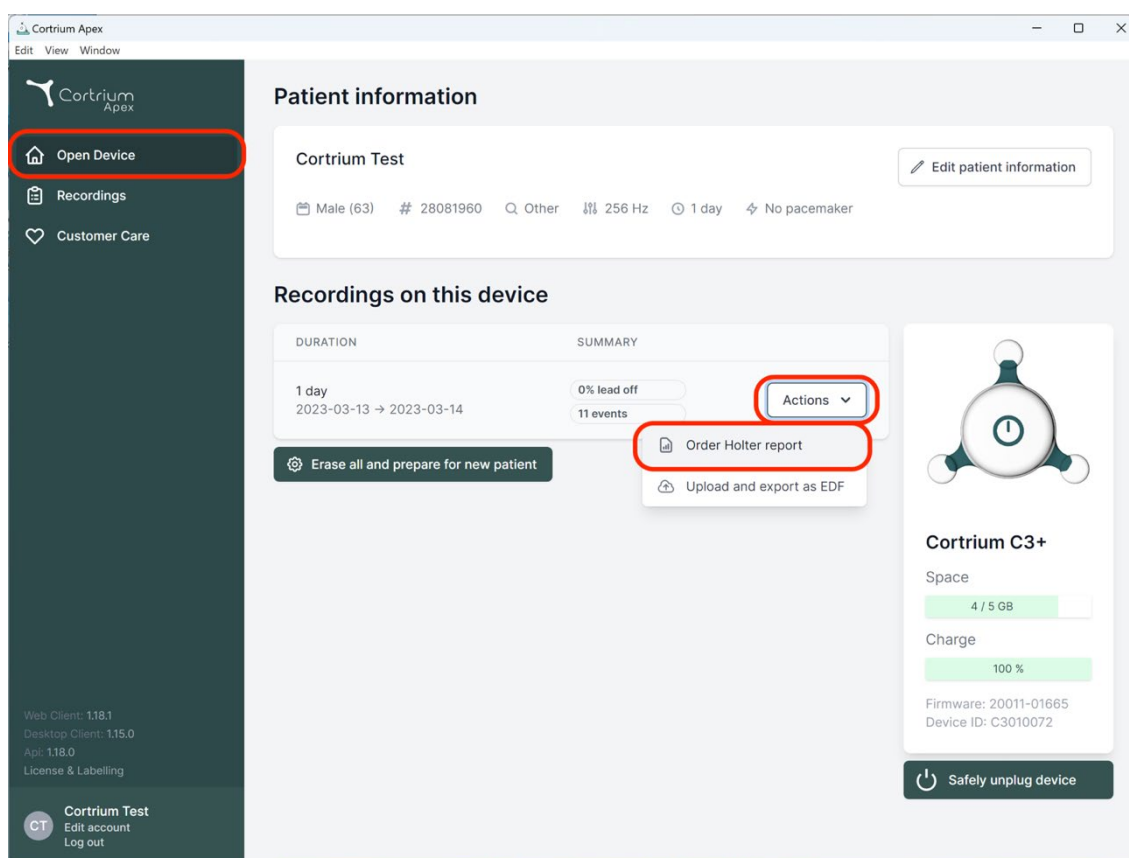
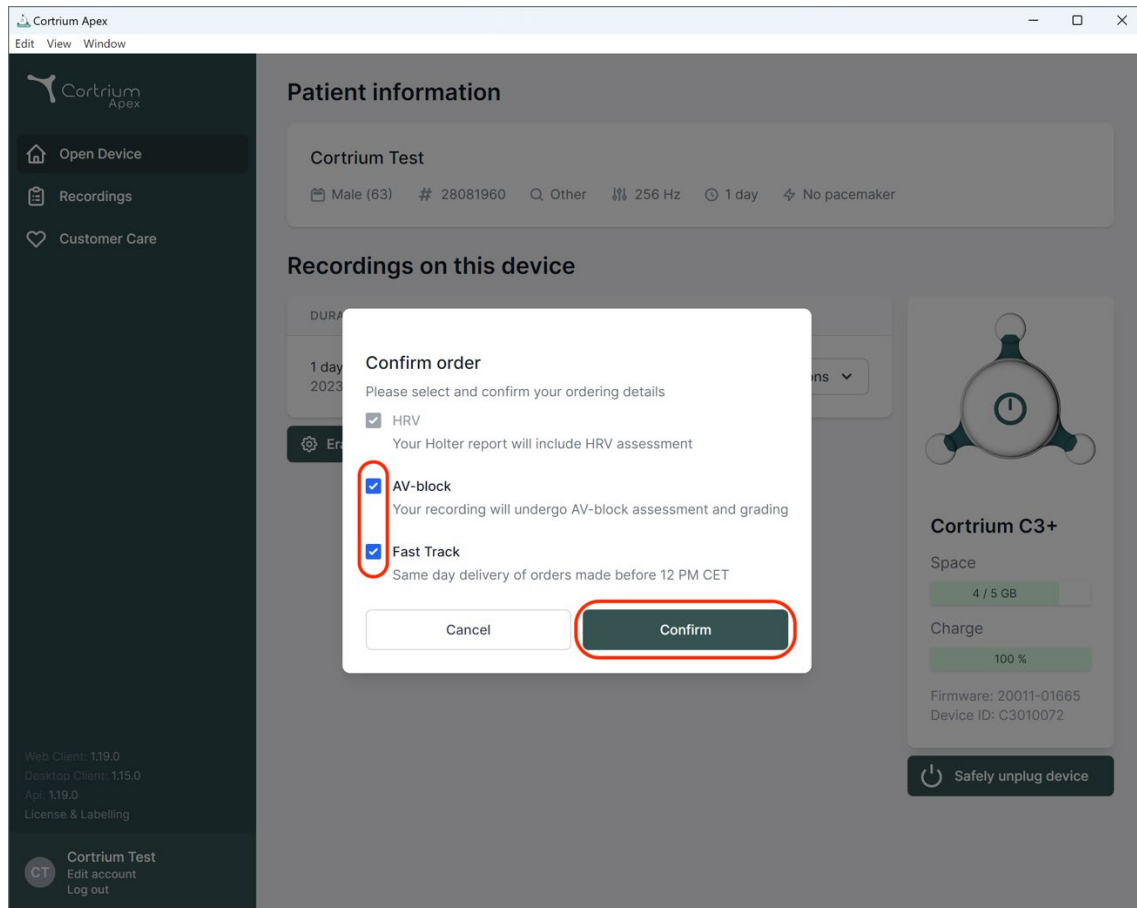
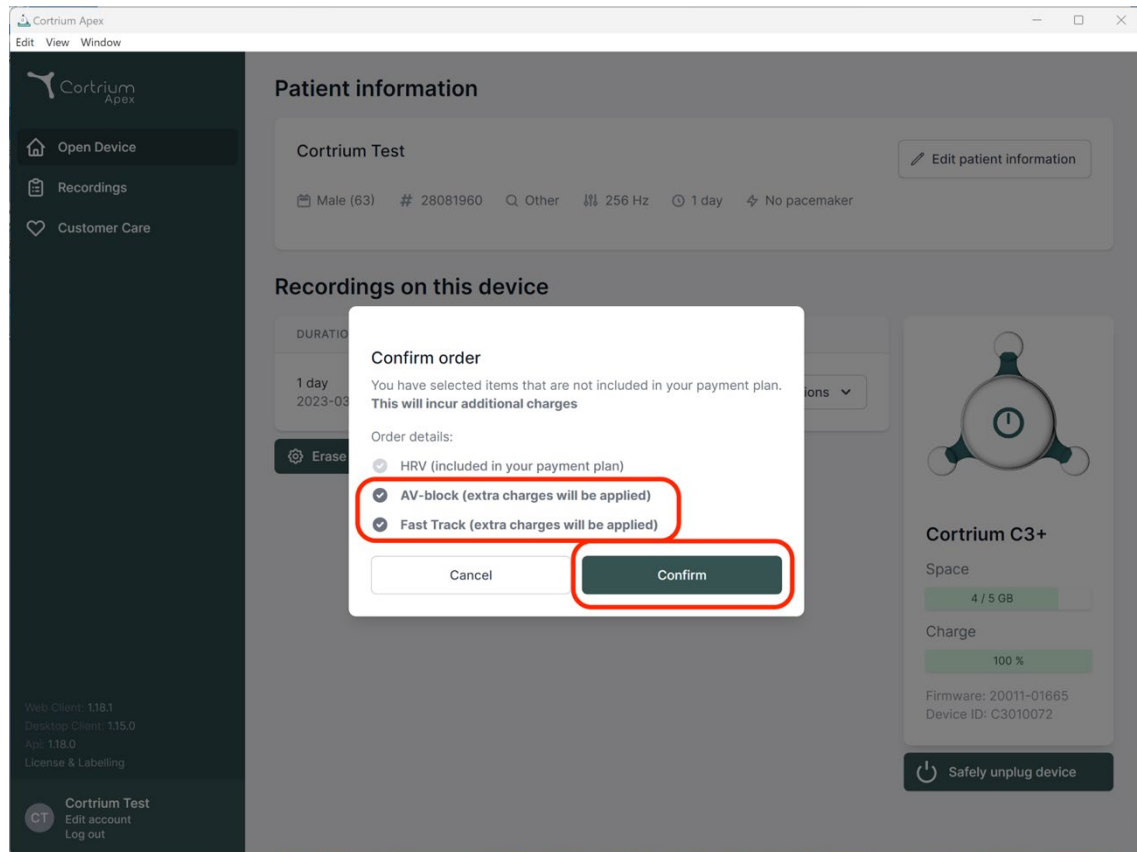
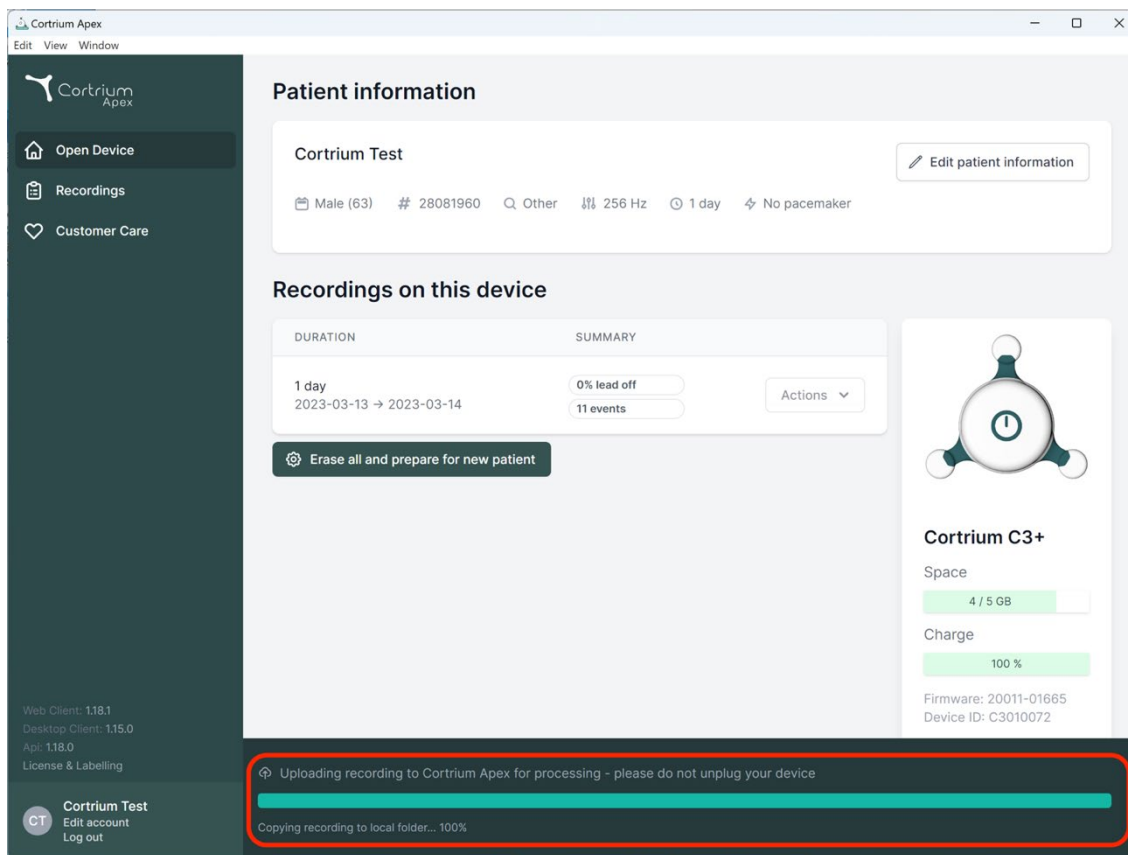
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are system status indicators: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'Api: 1.18.0', and 'License & Labelling'. At the bottom of the sidebar is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and other parameters like '256 Hz', '1 day', and 'No pacemaker'. An 'Edit patient information' button is present. Below this is the 'Recordings on this device' section, which shows a table with columns for 'DURATION' and 'SUMMARY'. A recording for '1 day' (2023-03-13 to 2023-03-14) is listed with '0% lead off' and '11 events'. An 'Actions' dropdown menu is next to it. A button 'Erase all and prepare for new patient' is also visible. To the right of the table is a device status card for 'Cortrium C3+' showing 'Space' (4 / 5 GB) and 'Charge' (100 %). It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. At the bottom of the interface, a red-bordered box highlights a status bar with the text 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and a progress bar. Below this, it says 'Copying recording to local folder... 100%'.

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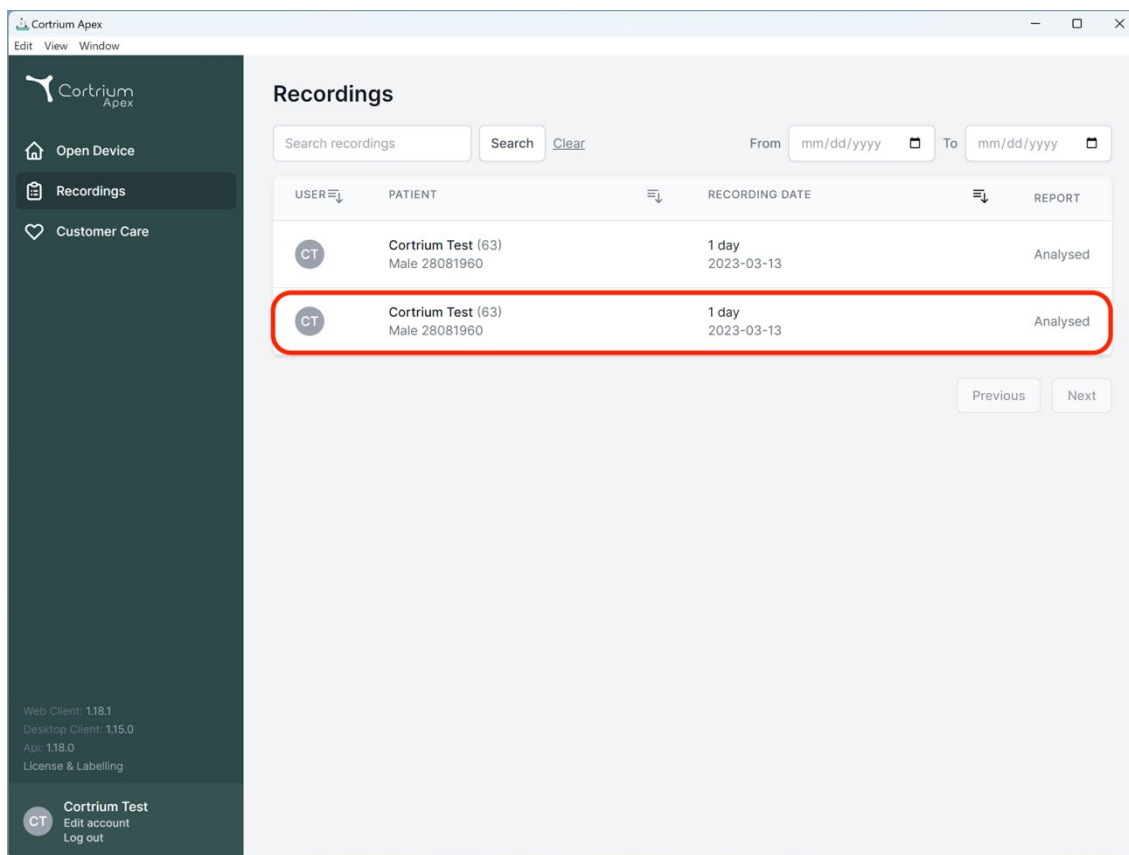
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. The interface includes a sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main area shows a table of recordings with the following data:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

The second row is highlighted with a red border. Below the table are 'Previous' and 'Next' navigation buttons. The bottom of the sidebar shows version information: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'App: 1.18.0', and 'License & Labelling'. The bottom right of the sidebar contains a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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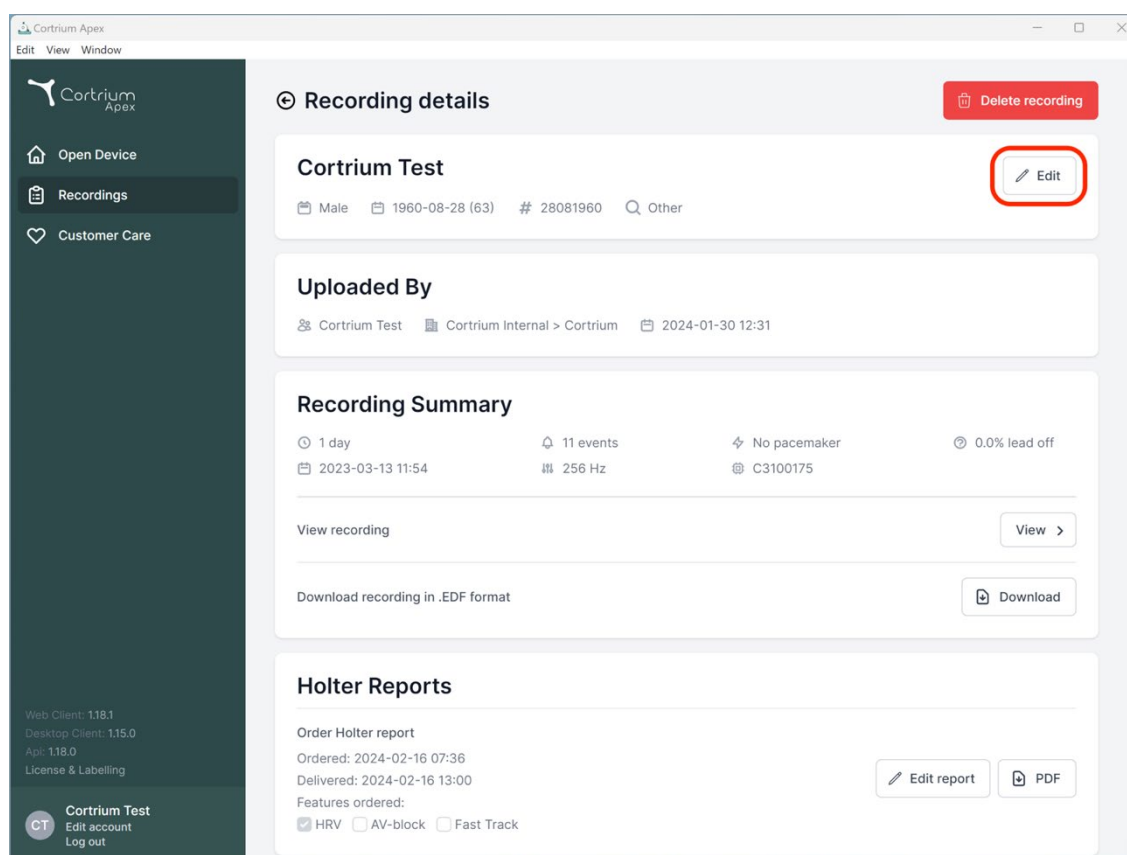
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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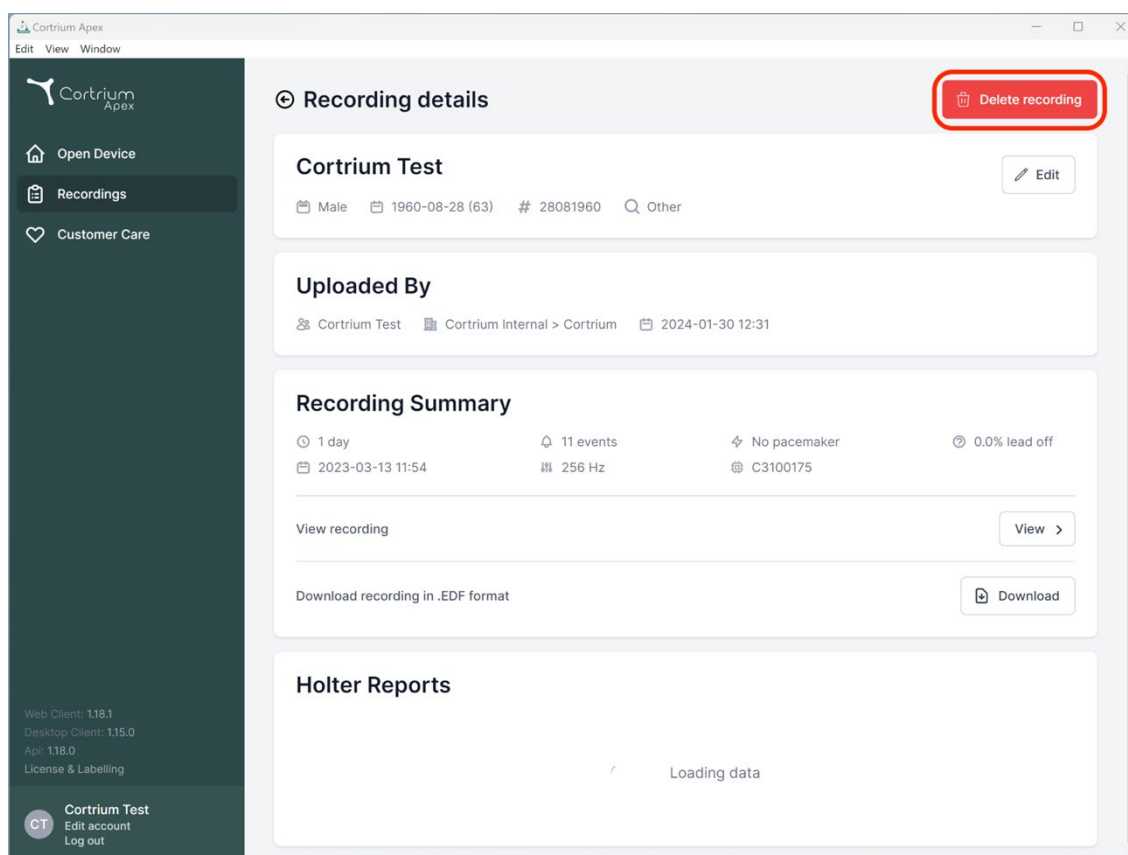
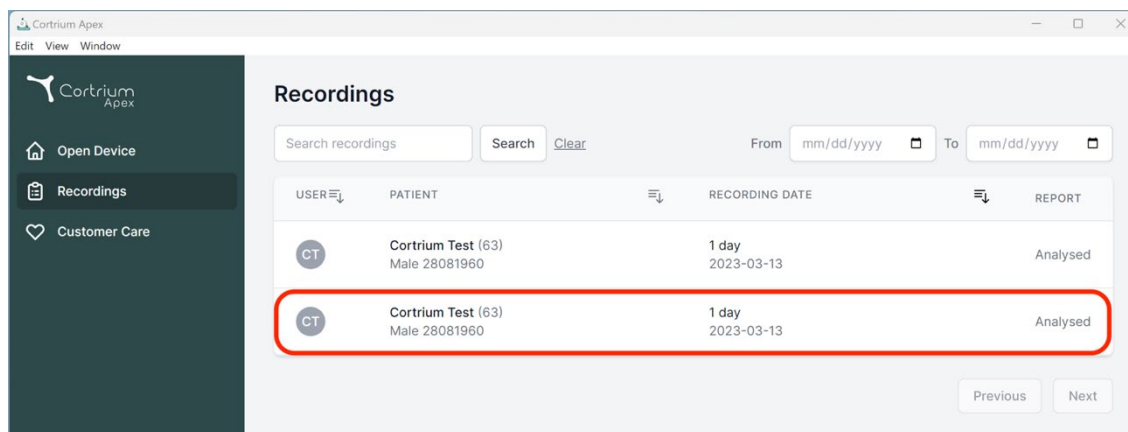
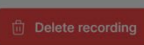
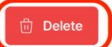
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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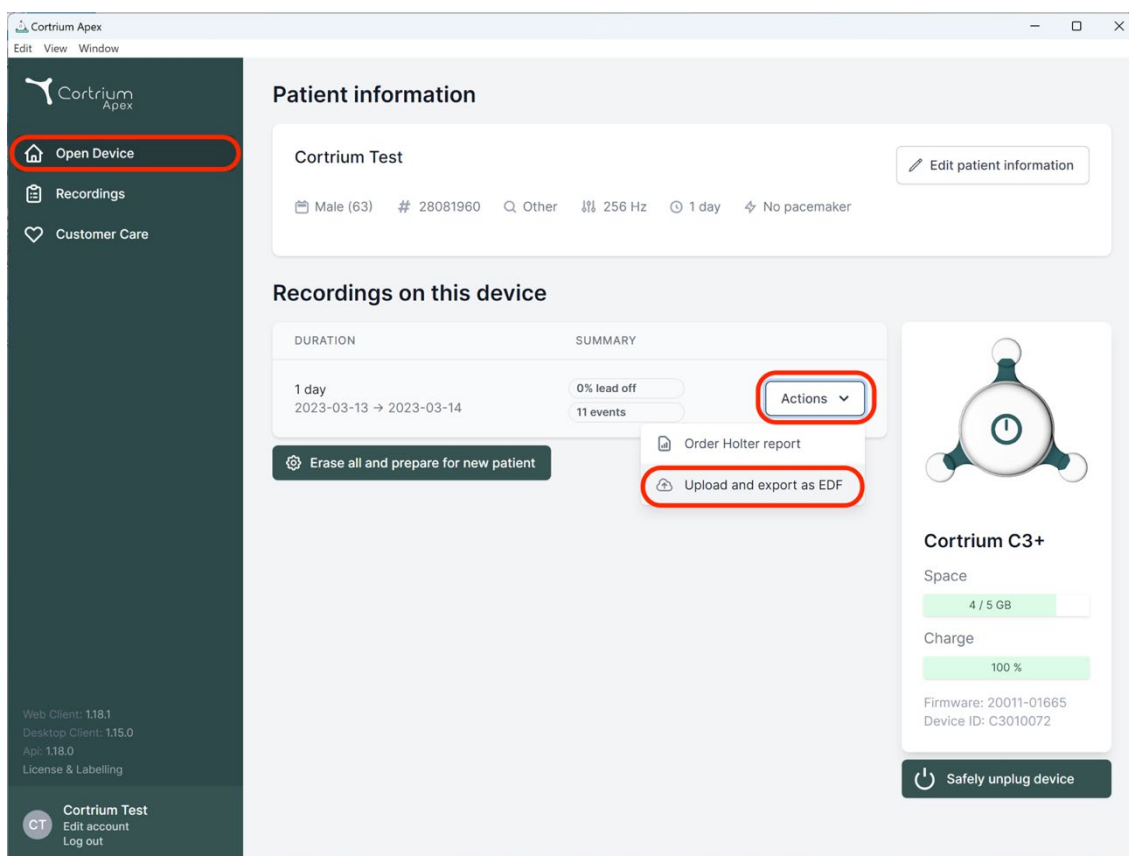
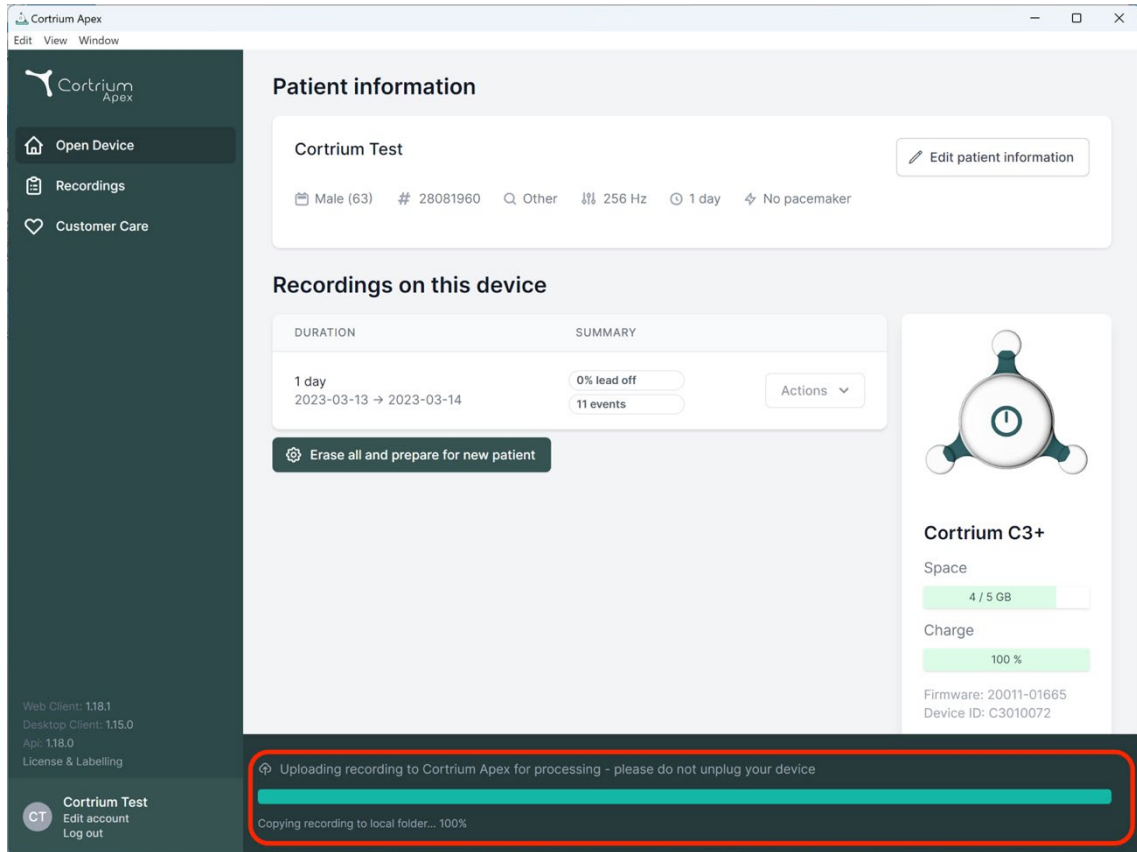
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

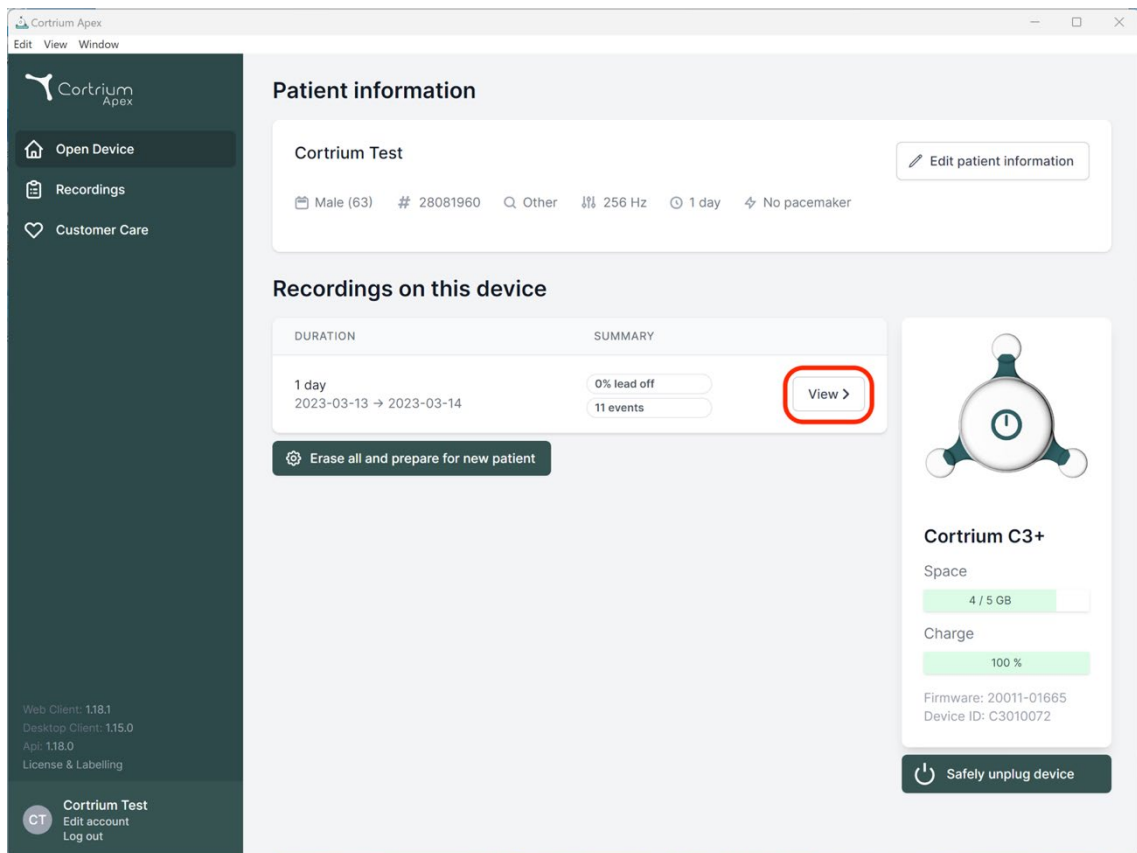
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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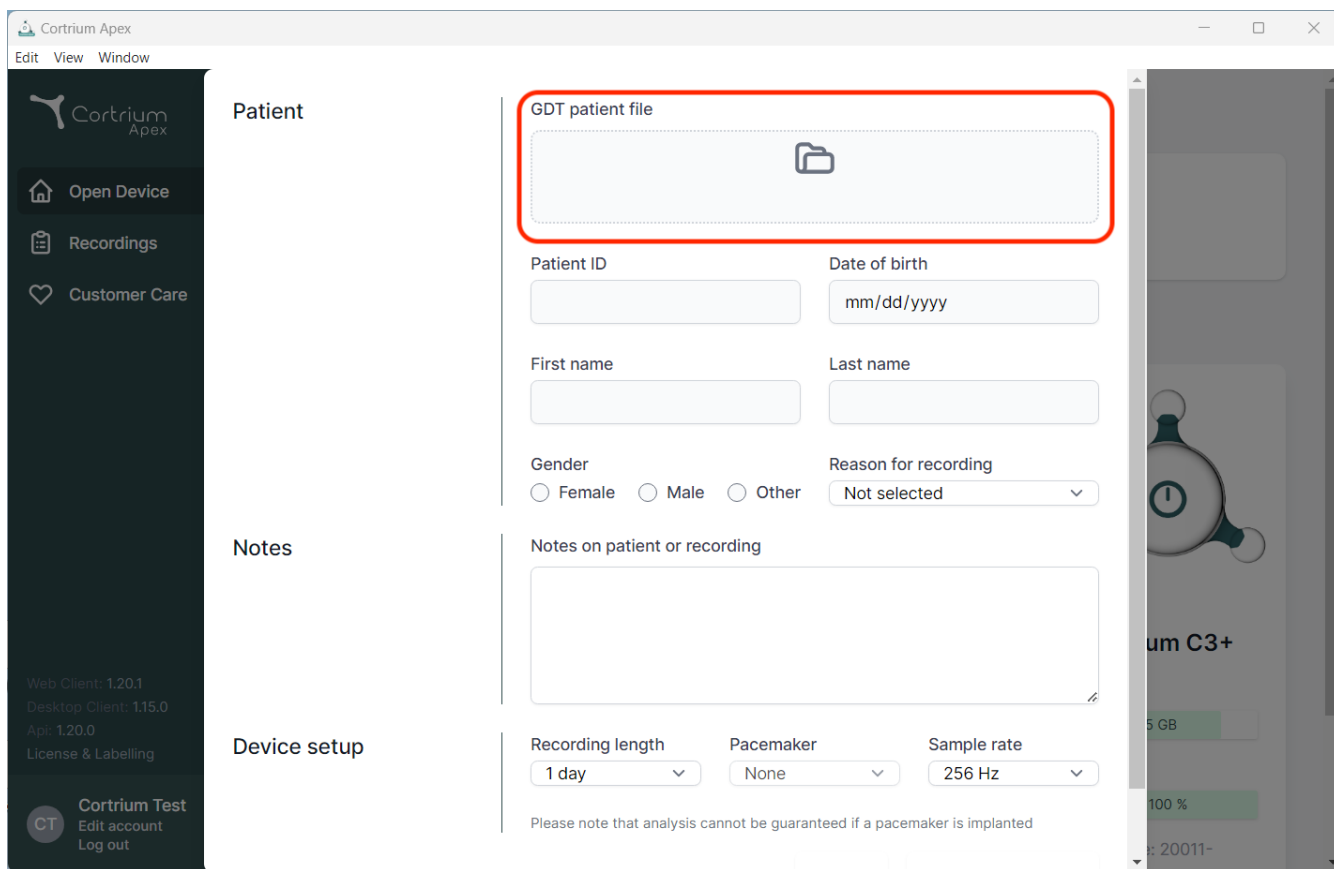
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". The main content area is divided into three sections: "Patient", "Notes", and "Device setup".

Patient section:

- GDT patient file:** A red-bordered box containing a folder icon, indicating where to upload the patient file.
- Patient ID:** A text input field.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

Notes section:

- Notes on patient or recording:** A large text area for entering notes.

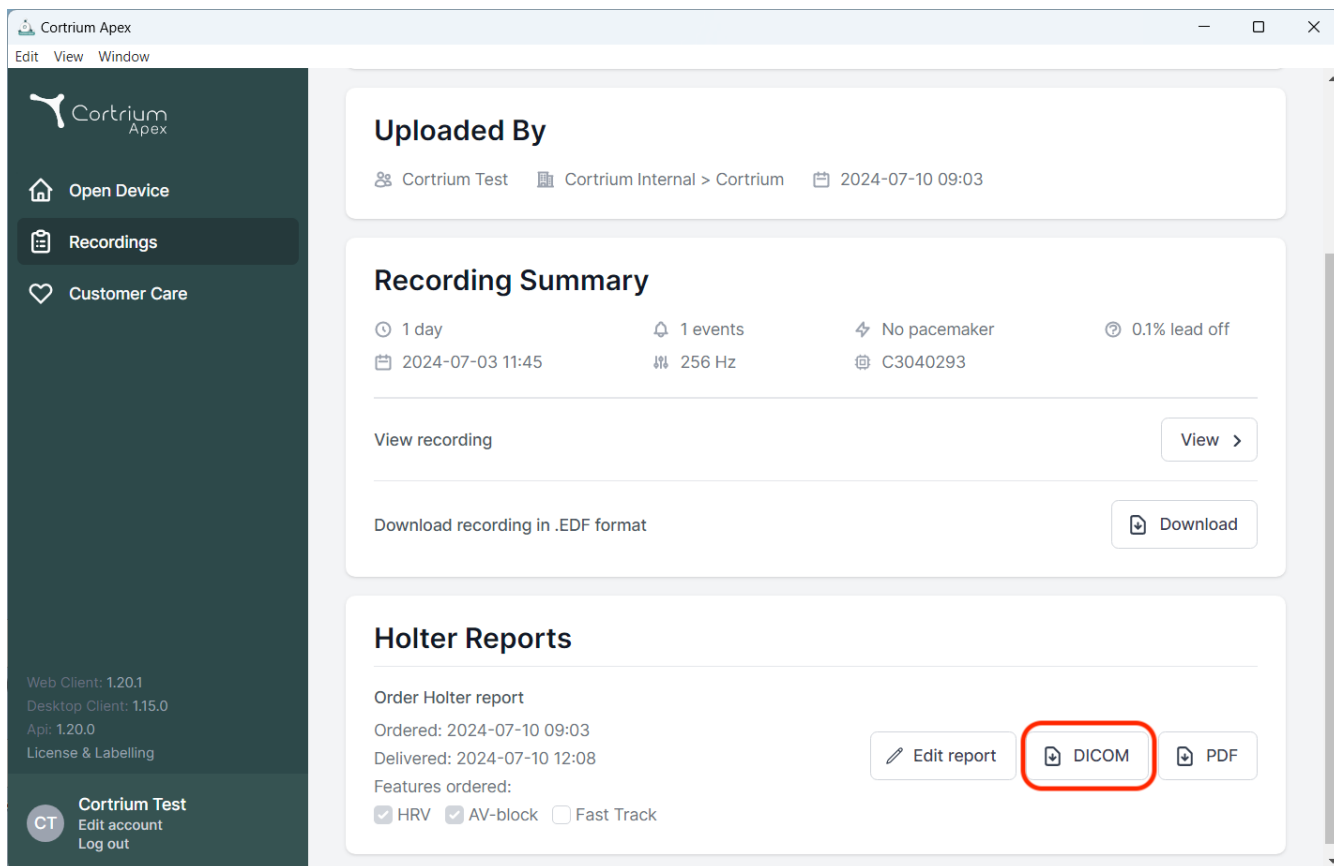
Device setup section:

- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

At the bottom of the "Device setup" section, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By' (showing 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'), 'Recording Summary' (displaying recording details like '1 day', '1 events', 'No pacemaker', and '0.1% lead off'), and 'Holter Reports'. In the 'Holter Reports' section, under 'Order Holter report', there are buttons for 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'. Below these buttons, the 'Features ordered' section shows checkboxes for 'HRV' (checked), 'AV-block' (checked), and 'Fast Track' (unchecked).

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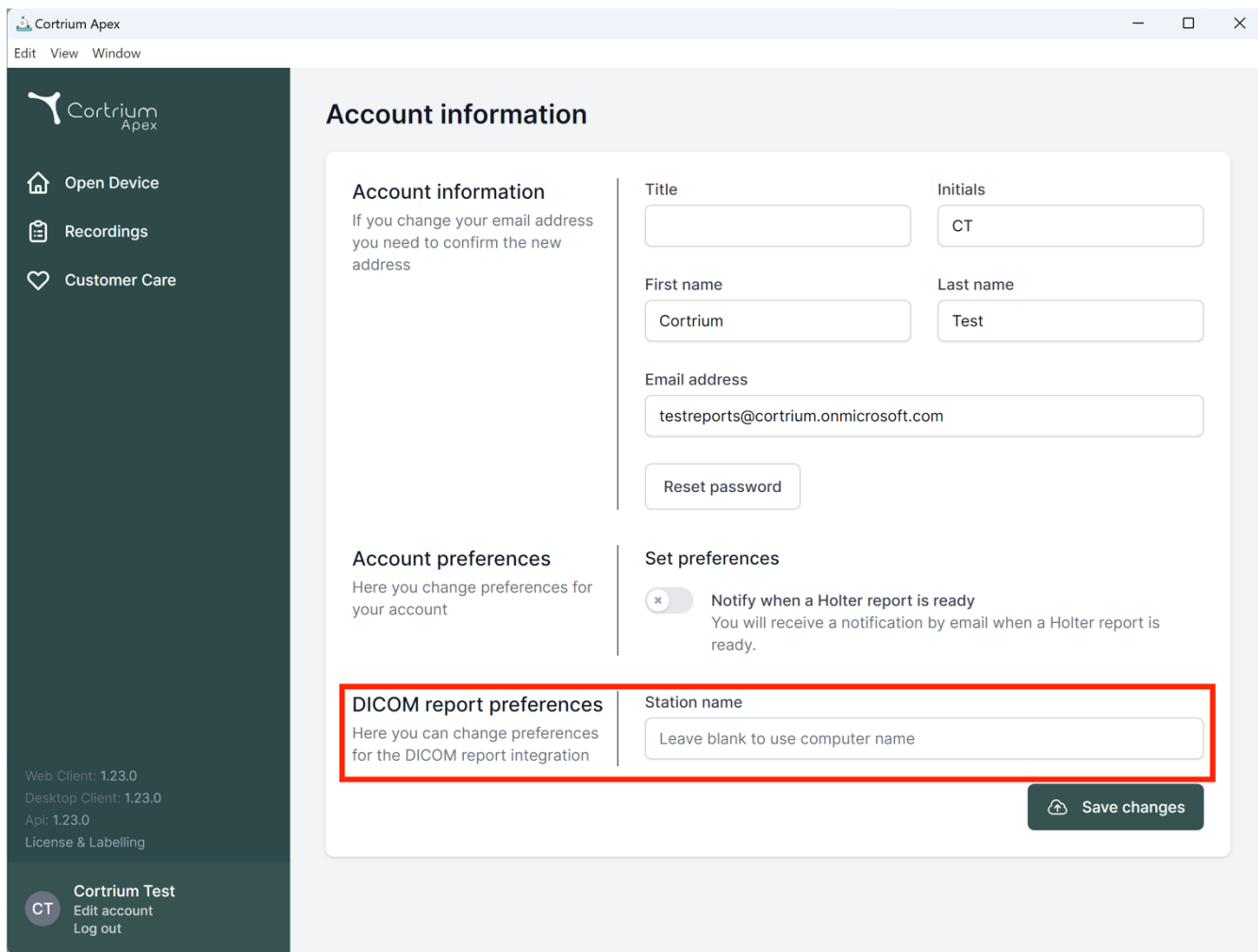The screenshot displays the 'Cortrium Apex' web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Account information' and is divided into three sections: 'Account information', 'Account preferences', and 'DICOM report preferences'. The 'Account information' section contains fields for Title, Initials, First name, Last name, and Email address, along with a 'Reset password' button. The 'Account preferences' section includes a toggle for 'Notify when a Holter report is ready'. The 'DICOM report preferences' section, which is highlighted with a red border, contains a 'Station name' field with the placeholder text 'Leave blank to use computer name'. A 'Save changes' button is located at the bottom right of the form.

Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

☒ **Set preferences**

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

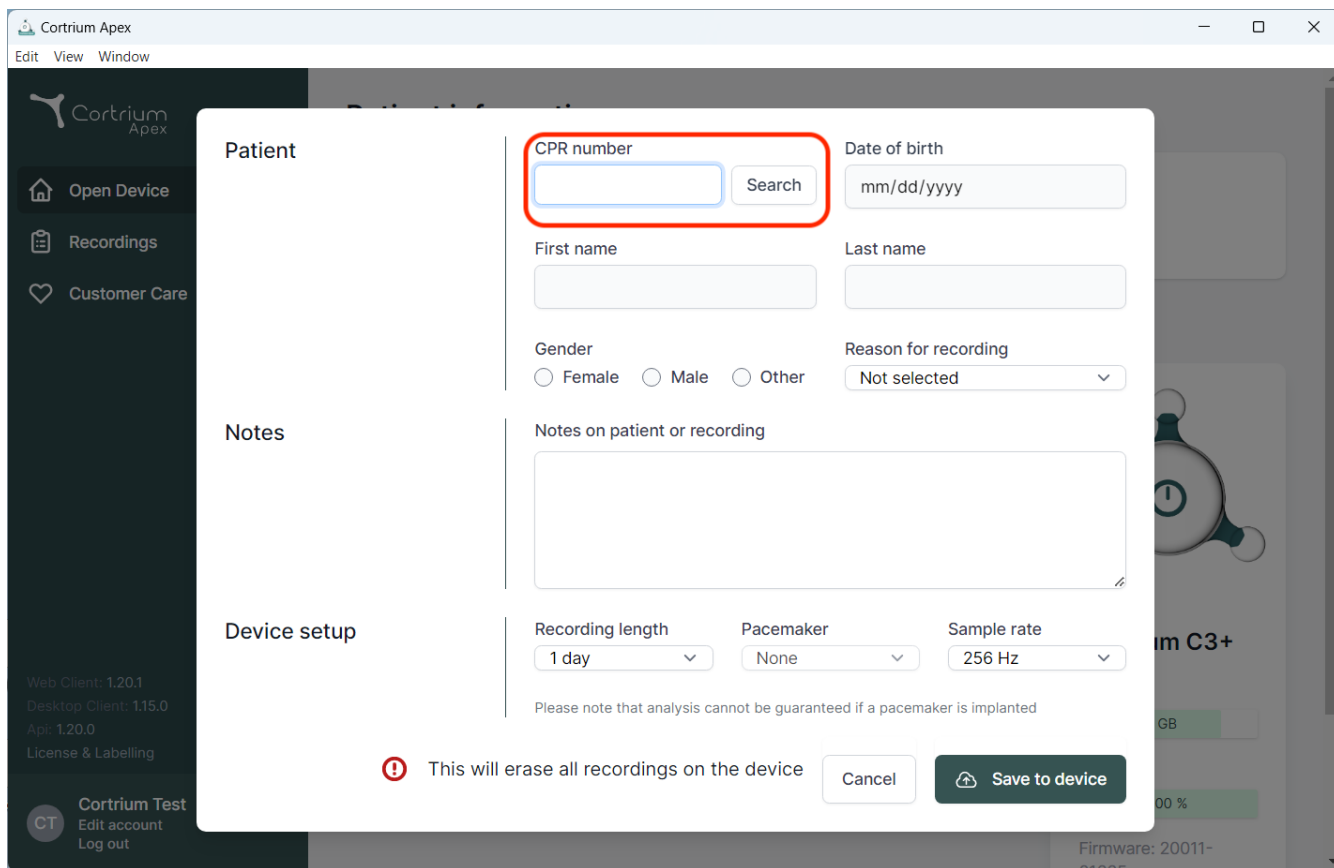
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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The screenshot shows the Cortrium Apex web application interface. A modal dialog titled "Patient" is open, allowing for patient search and data entry. The dialog is divided into three main sections: "Patient", "Notes", and "Device setup".

Patient Section:

- CPR number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangular box.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Three radio buttons labeled "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large, empty text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

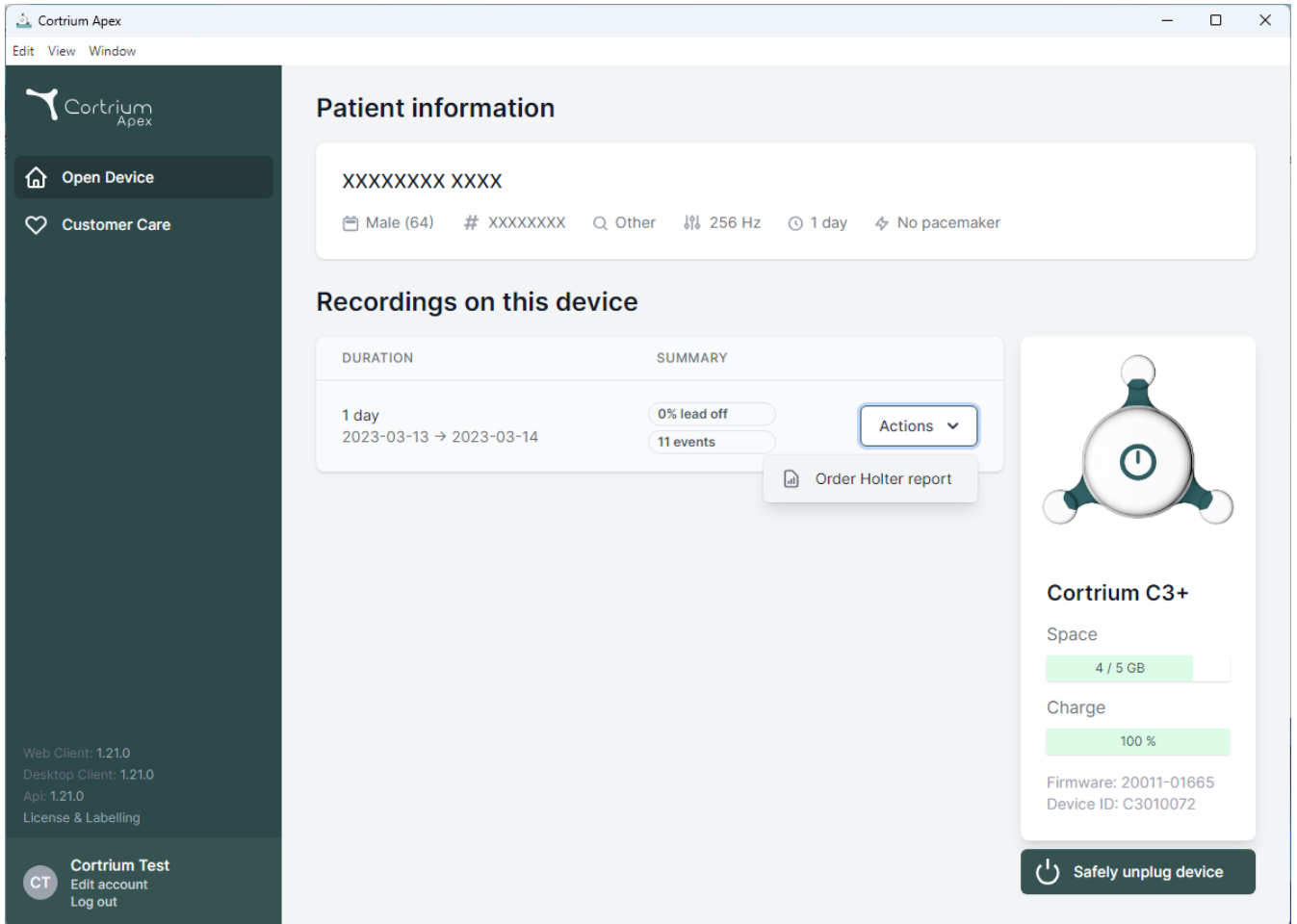
Below the device setup section, there is a warning message: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom of the dialog, there is a red warning icon followed by the text "This will erase all recordings on the device". To the right of this message are two buttons: "Cancel" and "Save to device".

The background of the application shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar contains version information (Web Client: 1.20.1, Desktop Client: 1.15.0, App: 1.20.0) and a user profile section for "Cortrium Test" with options to "Edit account" and "Log out".

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device', 'Customer Care', and version information (Web Client: 1.21.0, Desktop Client: 1.21.0, Api: 1.21.0, License & Labelling). The main content area is divided into sections: 'Patient information' showing a masked patient name and details like 'Male (64)', 'XXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'; 'Recordings on this device' with a table of recordings; and a device status panel for 'Cortrium C3+' showing space usage (4/5 GB), charge (100%), firmware (20011-01665), and device ID (C3010072). A 'Safely unplug device' button is at the bottom right.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

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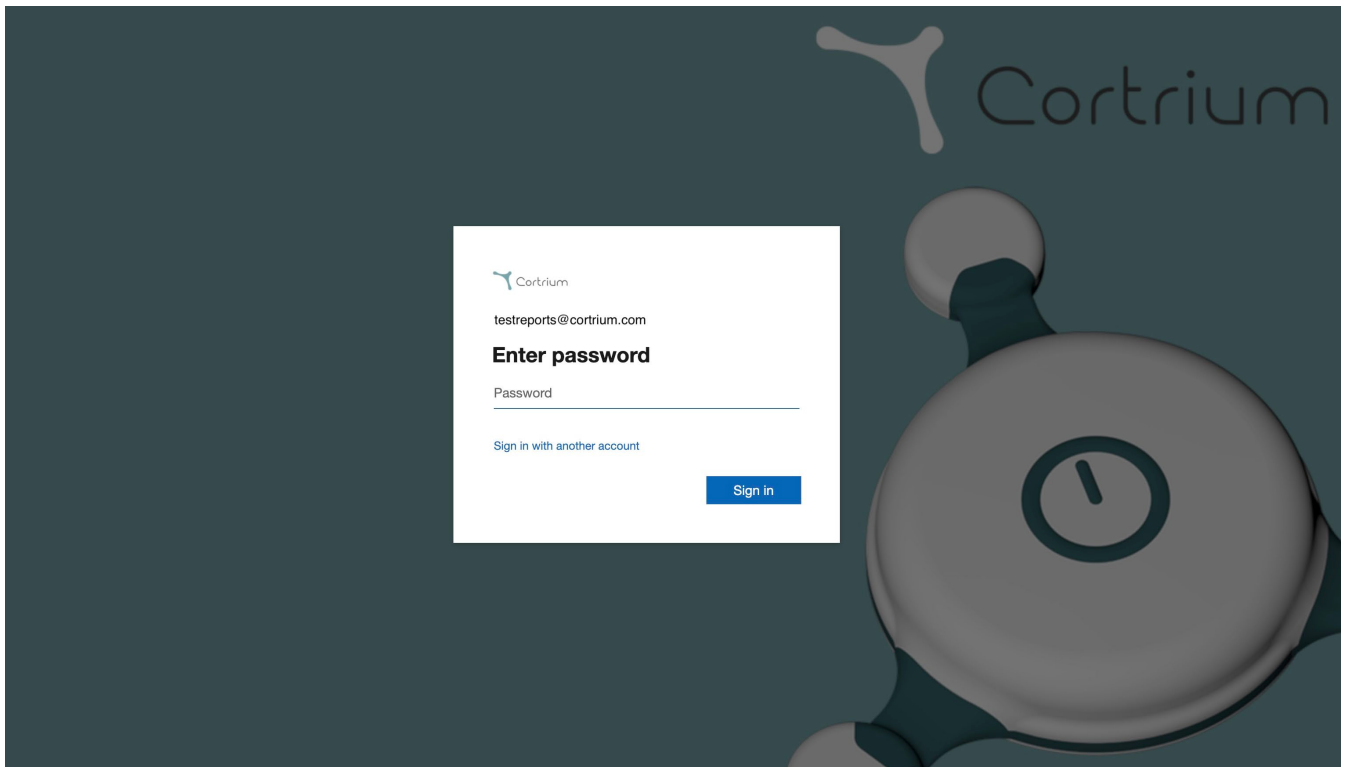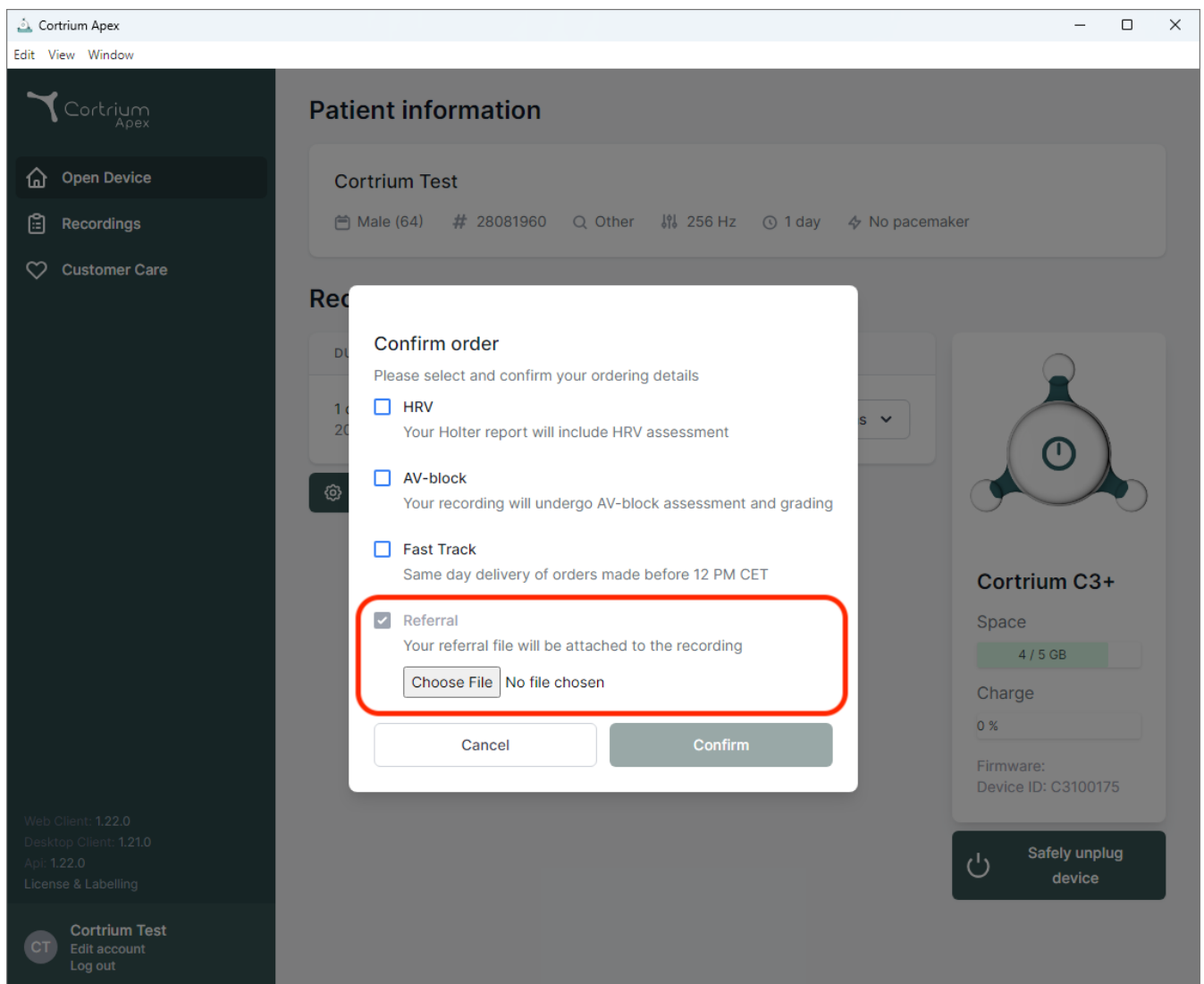
Test

User 03

Distributor 01 > Approver Organisation > Test Clinic

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
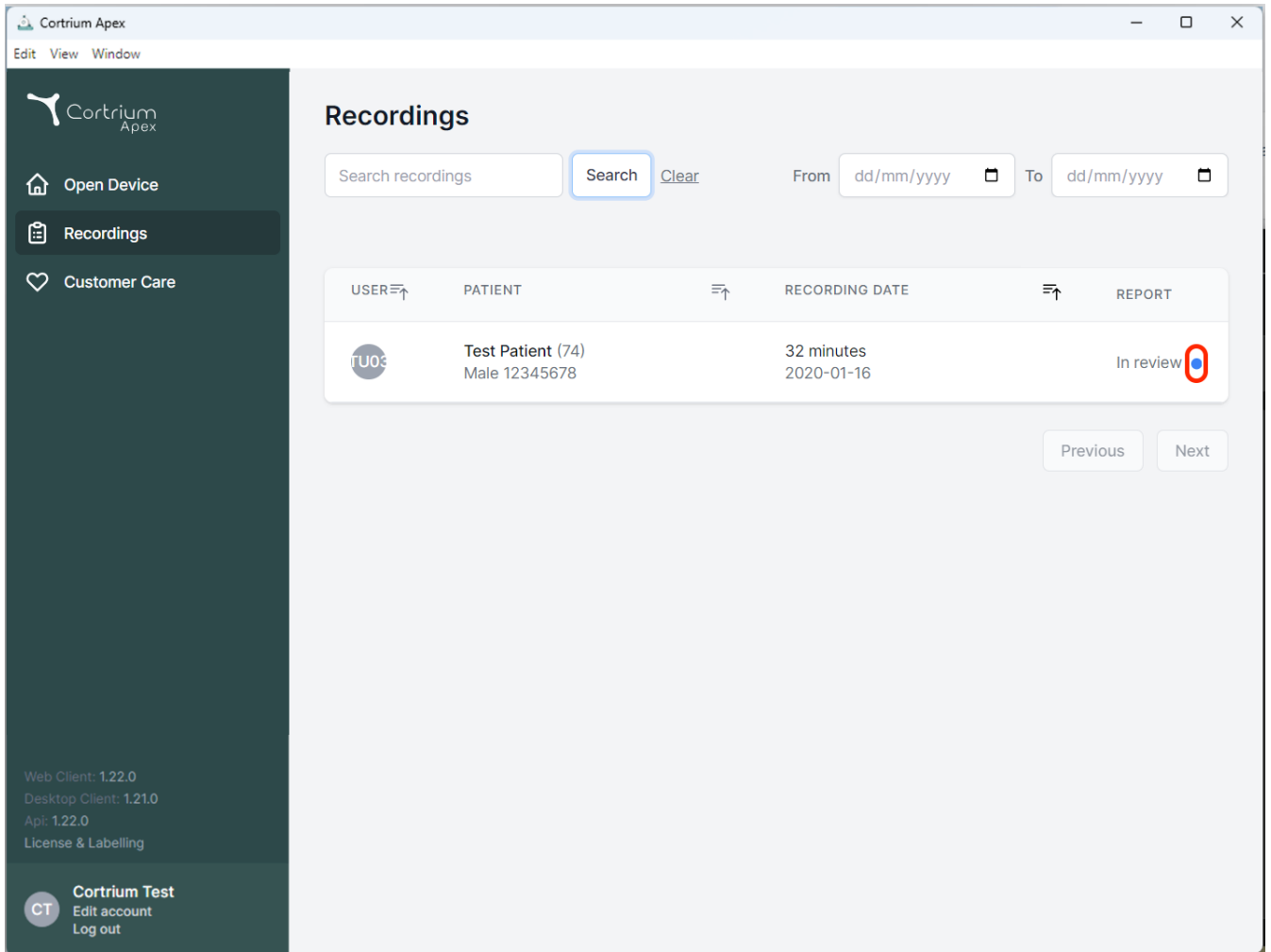
Search recordings [Clear](#)

From To

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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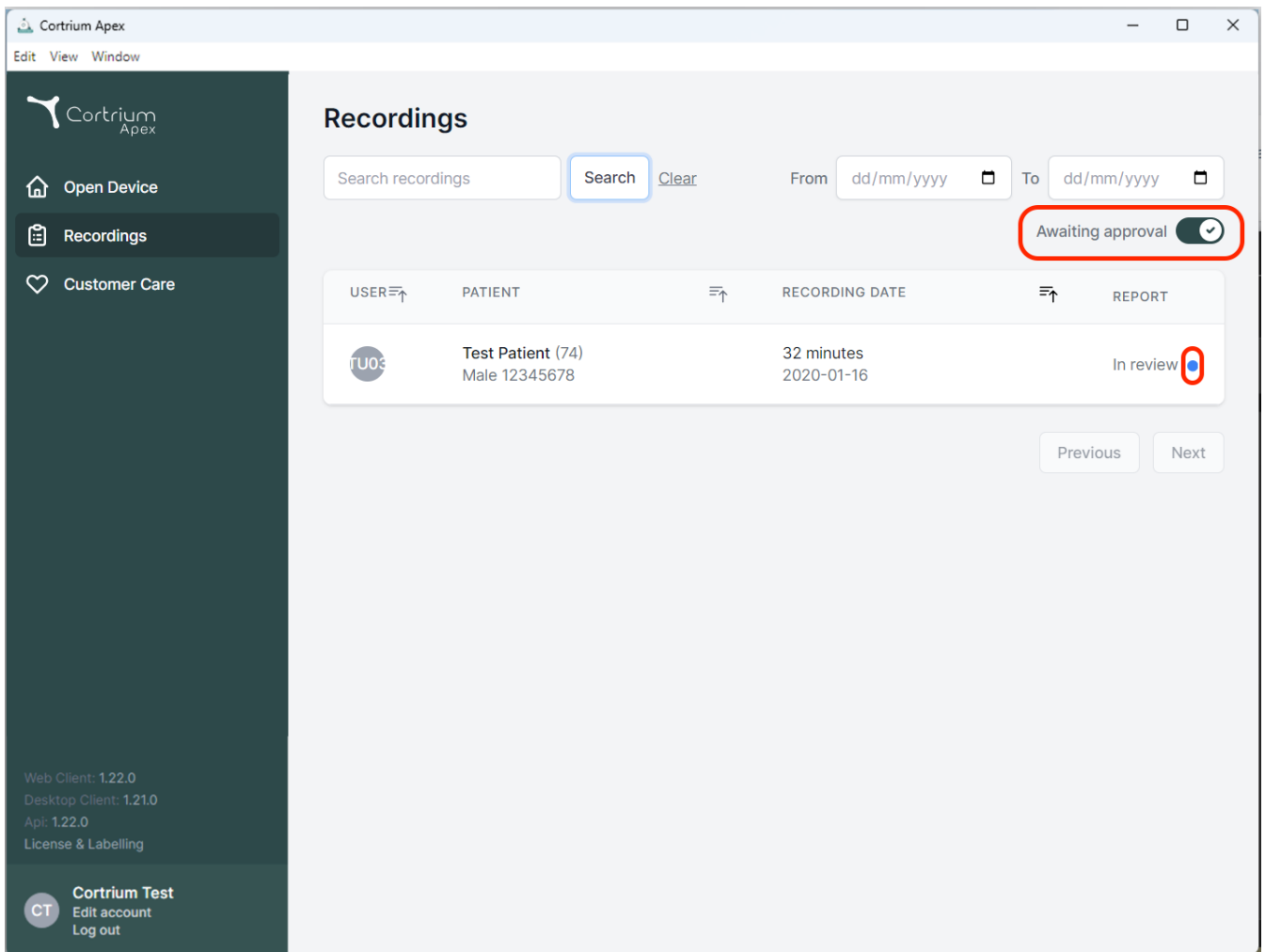Recordings

Search recordings From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
U03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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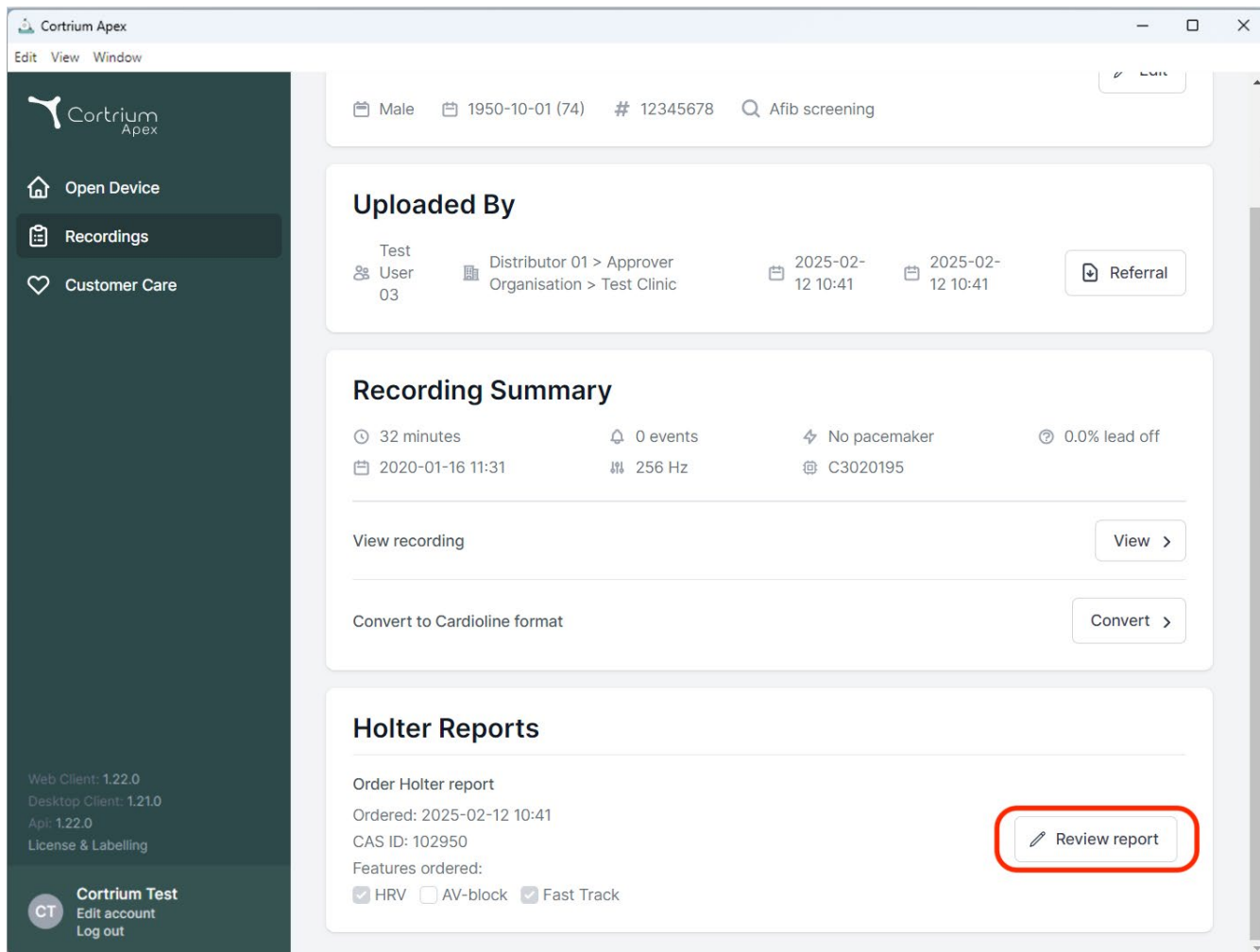
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section shows 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates of 2025-02-12 10:41. A 'Referral' button is present. The 'Recording Summary' section lists: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section shows 'Order Holter report' with 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

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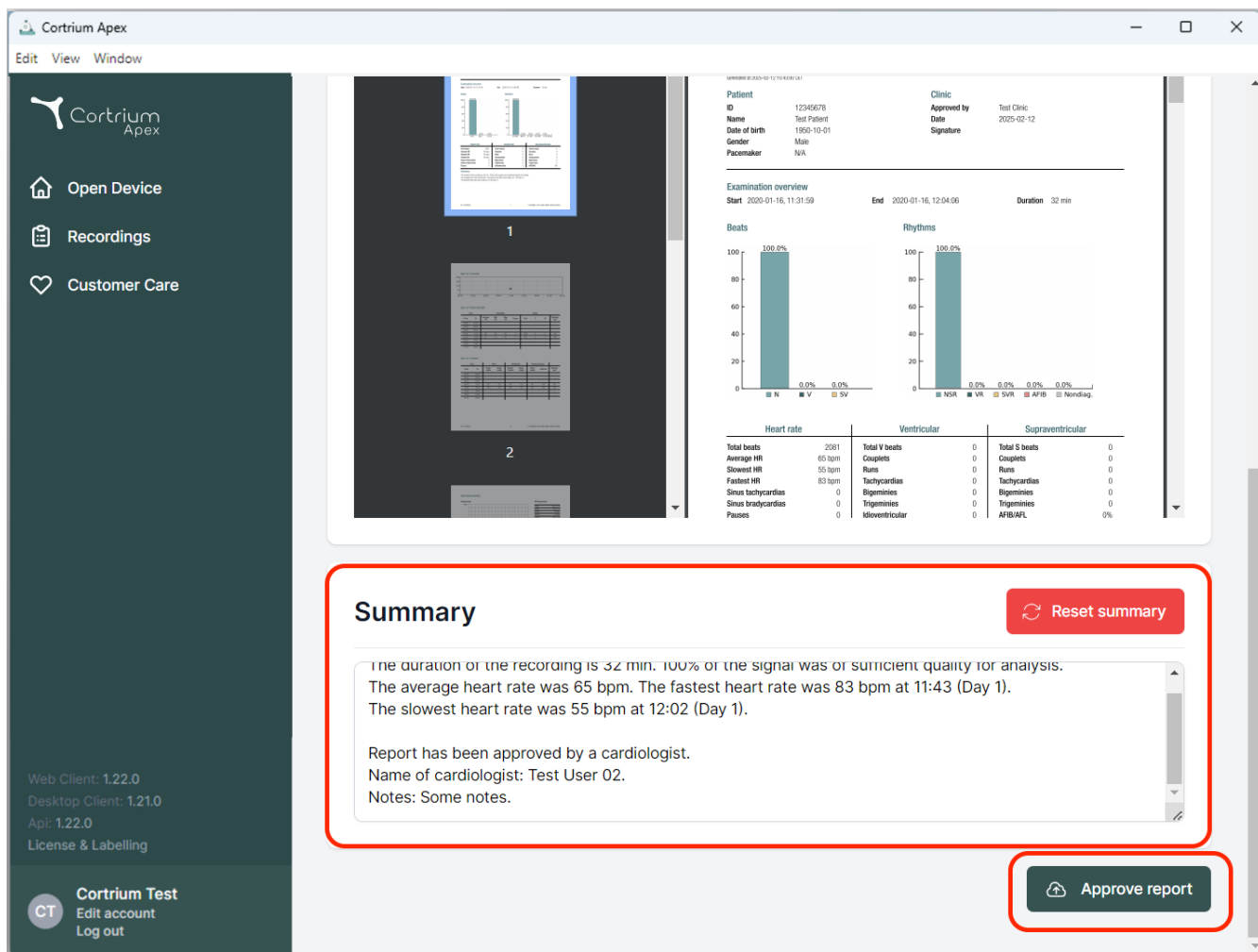Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

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