# Cortrium Apex 1.27

# Cortrium Apex 1.27

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
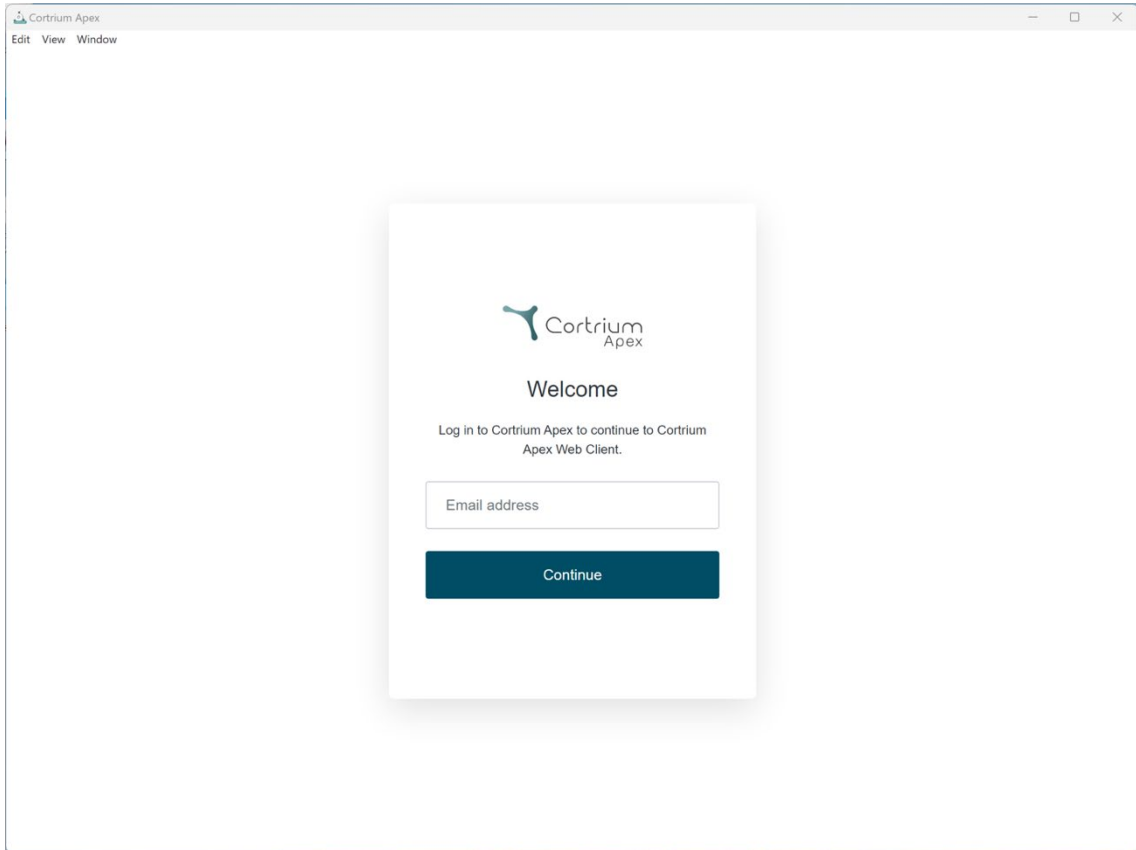
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". In the center of the browser is a white login dialog box with a subtle shadow. The dialog box features the Cortrium Apex logo at the top, followed by the heading "Welcome". Below the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." There is a text input field labeled "Email address" and a dark blue button labeled "Continue" at the bottom of the dialog.

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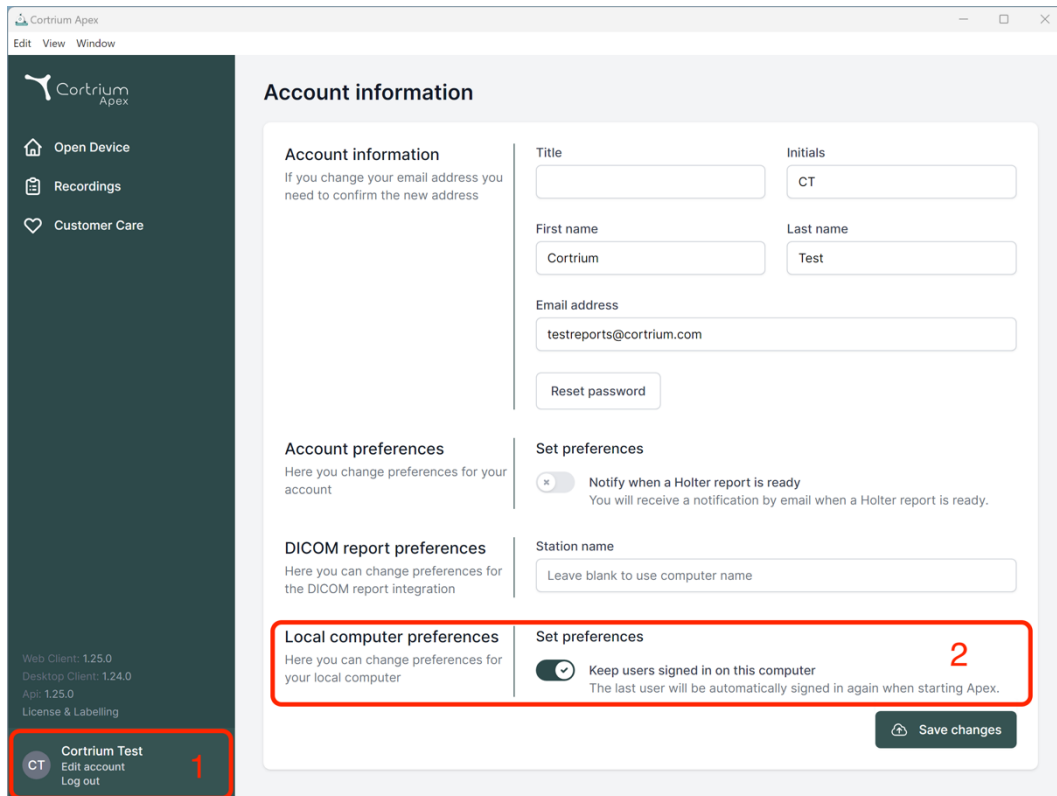**Local computer preferences**

Here you can change preferences for your local computer

**Set preferences**

☒ **Notify when a Holter report is ready**  
You will receive a notification by email when a Holter report is ready.

**Station name**

**Local computer preferences**

☒ **Keep users signed in on this computer**  
The last user will be automatically signed in again when starting Apex.

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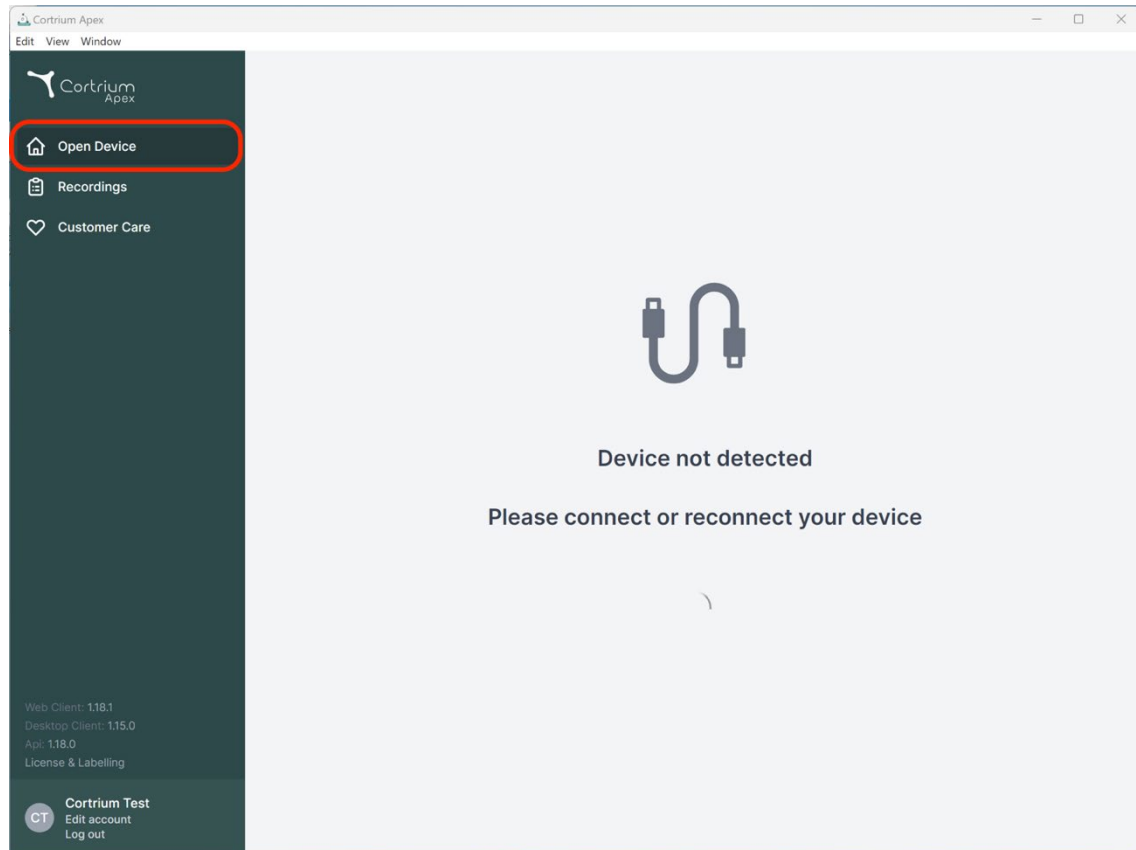
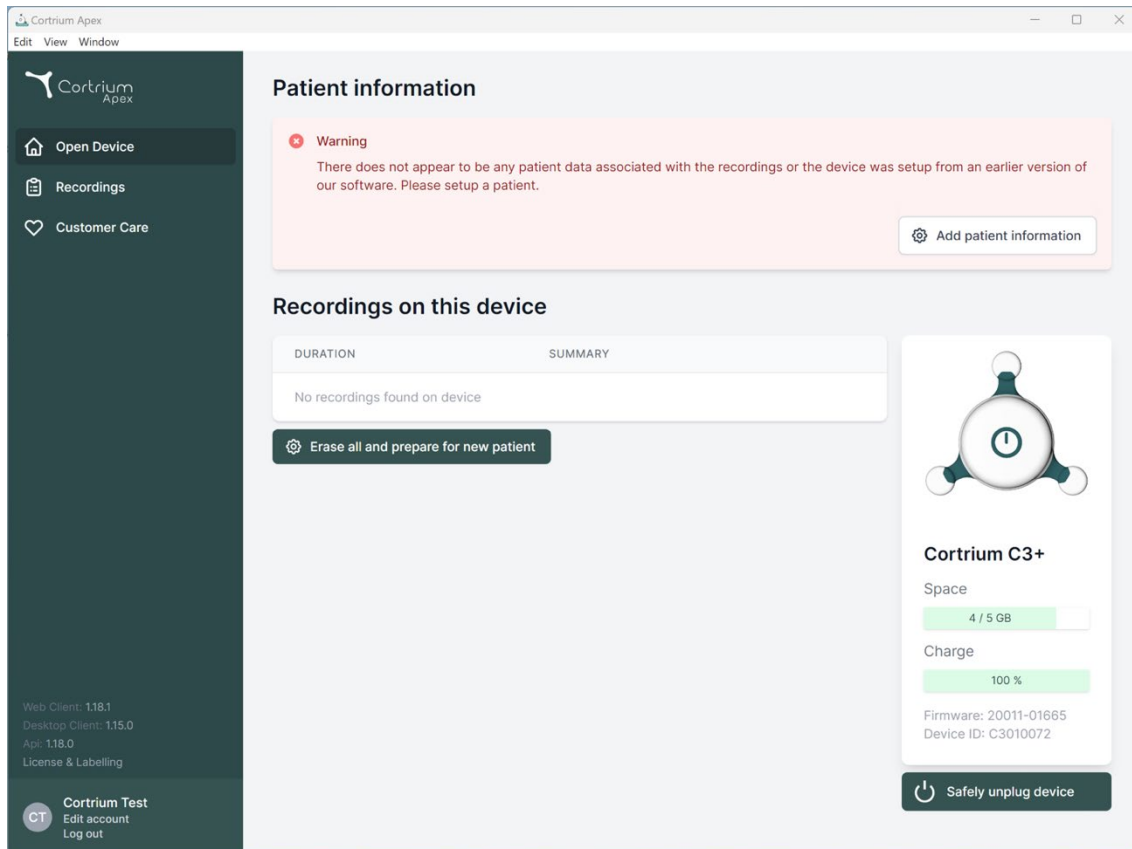
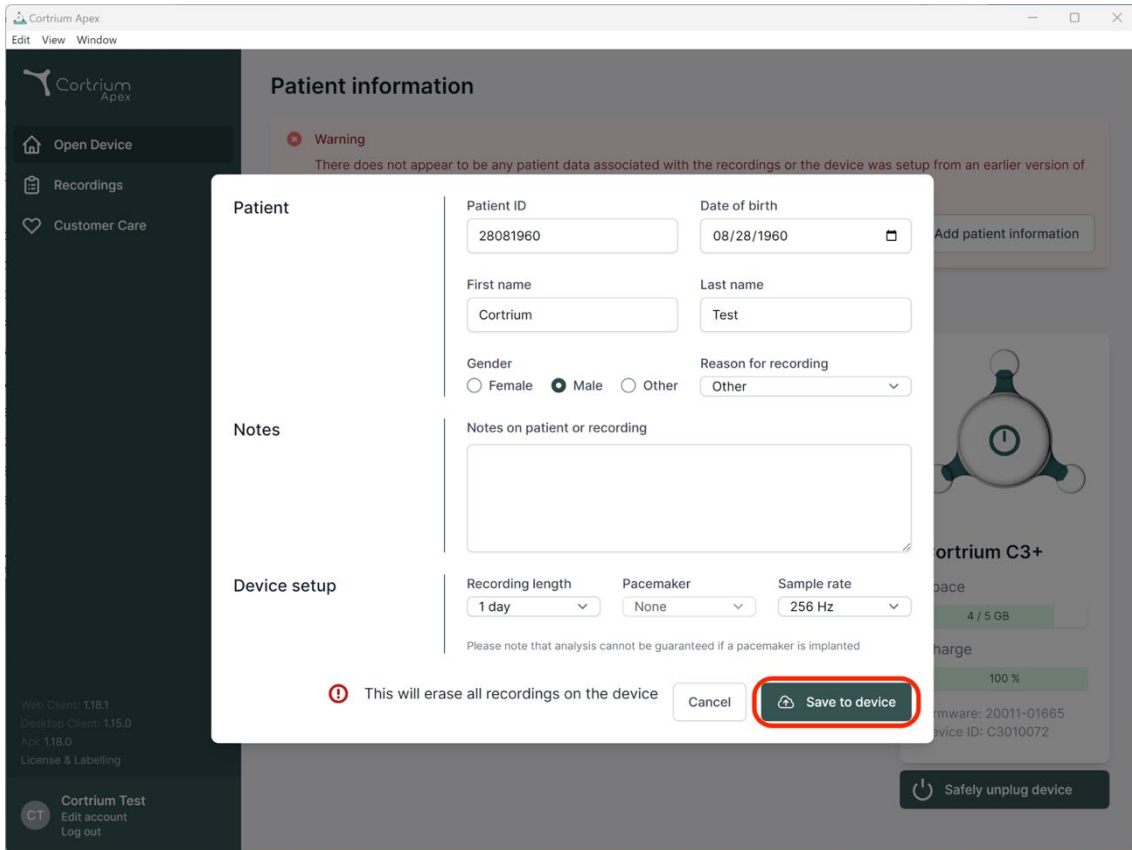
**Patient information**

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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Order Holter report

Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
☒ HRV ☐ AV-block ☐ Fast Track

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
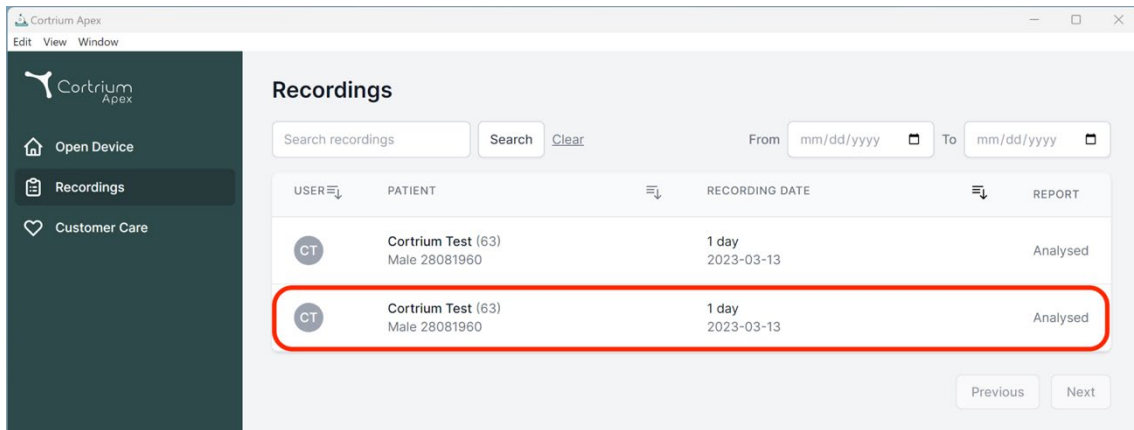
**Recordings**

Search recordings  Search [Clear](#) From  To

| USER | PATIENT                             | RECORDING DATE      | REPORT   |
|------|-------------------------------------|---------------------|----------|
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |
| CT   | Cortrium Test (63)<br>Male 28081960 | 1 day<br>2023-03-13 | Analysed |

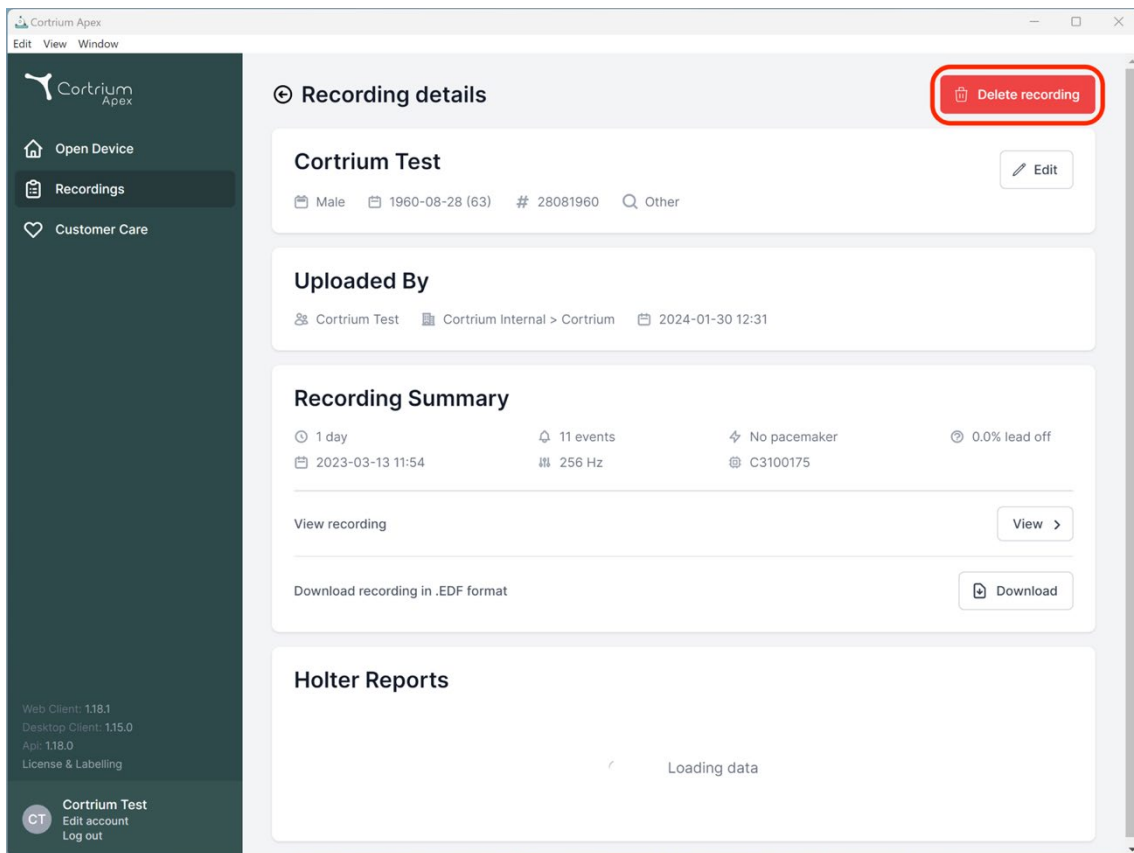Previous Next

**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

**Holter Reports**

Loading data

Web Client: 1.18.1  
Desktop Client: 1.15.0  
API: 1.18.0  
License & Labelling

CT Cortrium Test  
Edit account  
Log out

Cortrium Apex  
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

 Customer Care

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

 Cortrium Test  
Edit account  
Log out

### Recording details

#### Cortrium Test

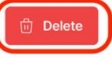
 Male  1960-08-28 (63)  # 28081960  Other 

#### Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

**Confirm delete**  
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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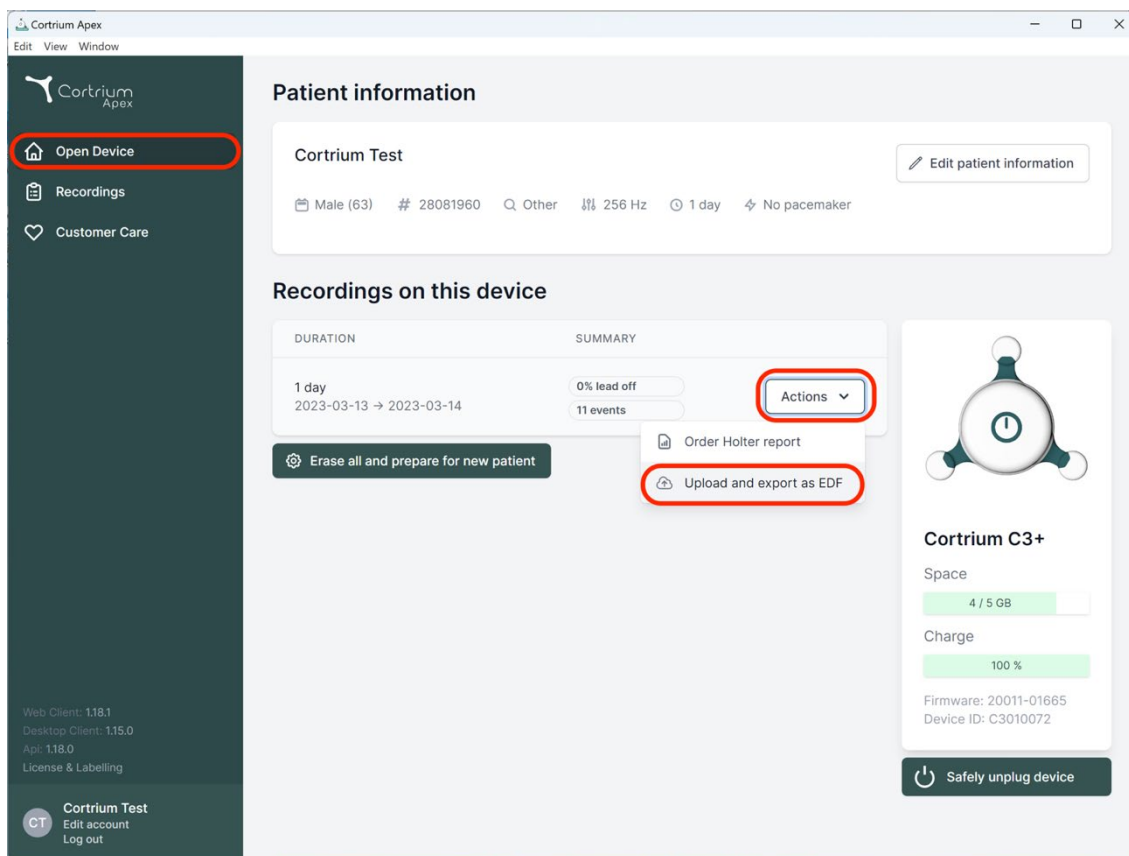
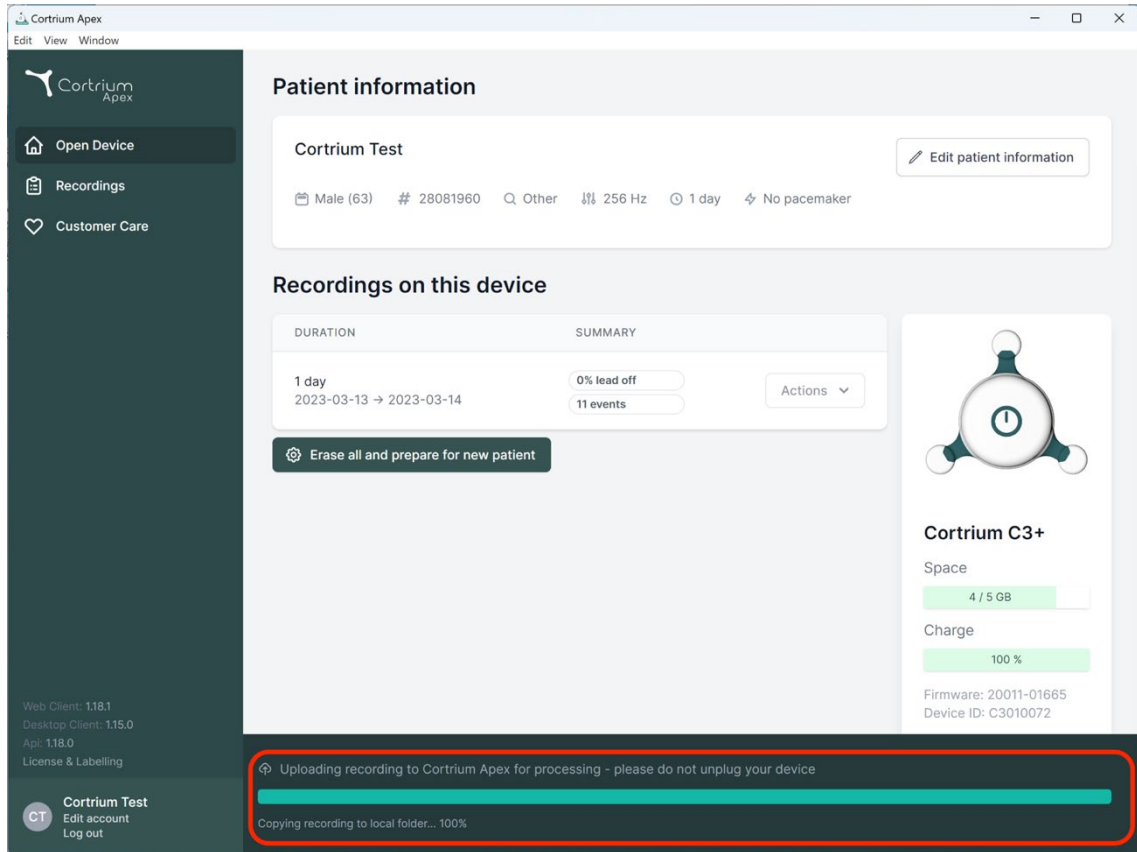
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  |                         |
|----------------------------------|--------------------------|-------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events | <a href="#">Actions</a> |

[Erase all and prepare for new patient](#)

**Cortrium C3+**

Space: 4 / 5 GB

Charge: 100 %

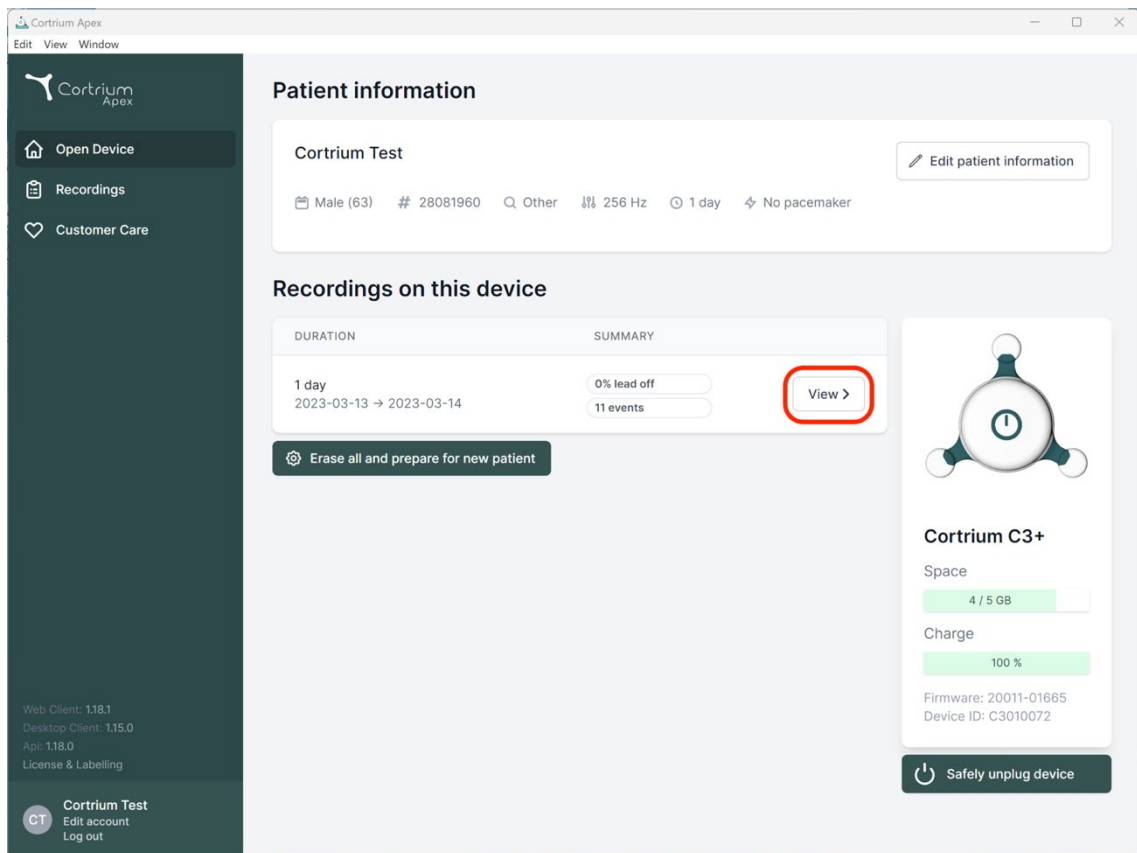
Firmware: 20011-01665  
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

**Cortrium Test**  
Edit account  
Log out

**Cortrium Apex**  
Edit View Window

**Patient information**

**Cortrium Test** [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  |                           |
|----------------------------------|--------------------------|---------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events | <a href="#">View &gt;</a> |

[Erase all and prepare for new patient](#)

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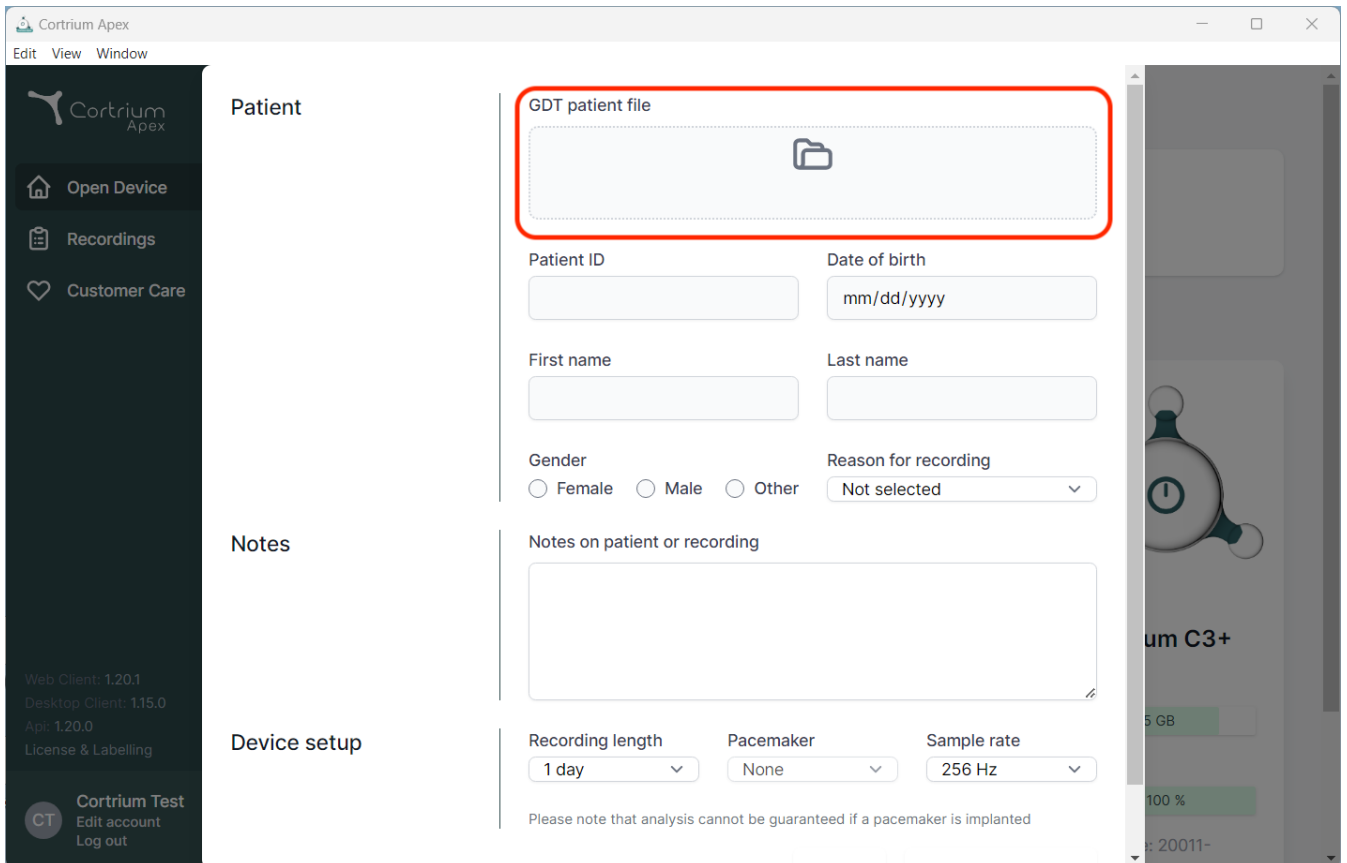
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", and "Api: 1.20.0". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.


The main content area is divided into three sections:

- Patient:** This section contains a "GDT patient file" upload area, which is highlighted with a red rectangle. Below this are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected").
- Notes:** This section has a text area labeled "Notes on patient or recording".
- Device setup:** This section includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below these fields states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device model, "Cortrium C3+", is visible, showing a battery level of "100 %" and a storage capacity of "5 GB".

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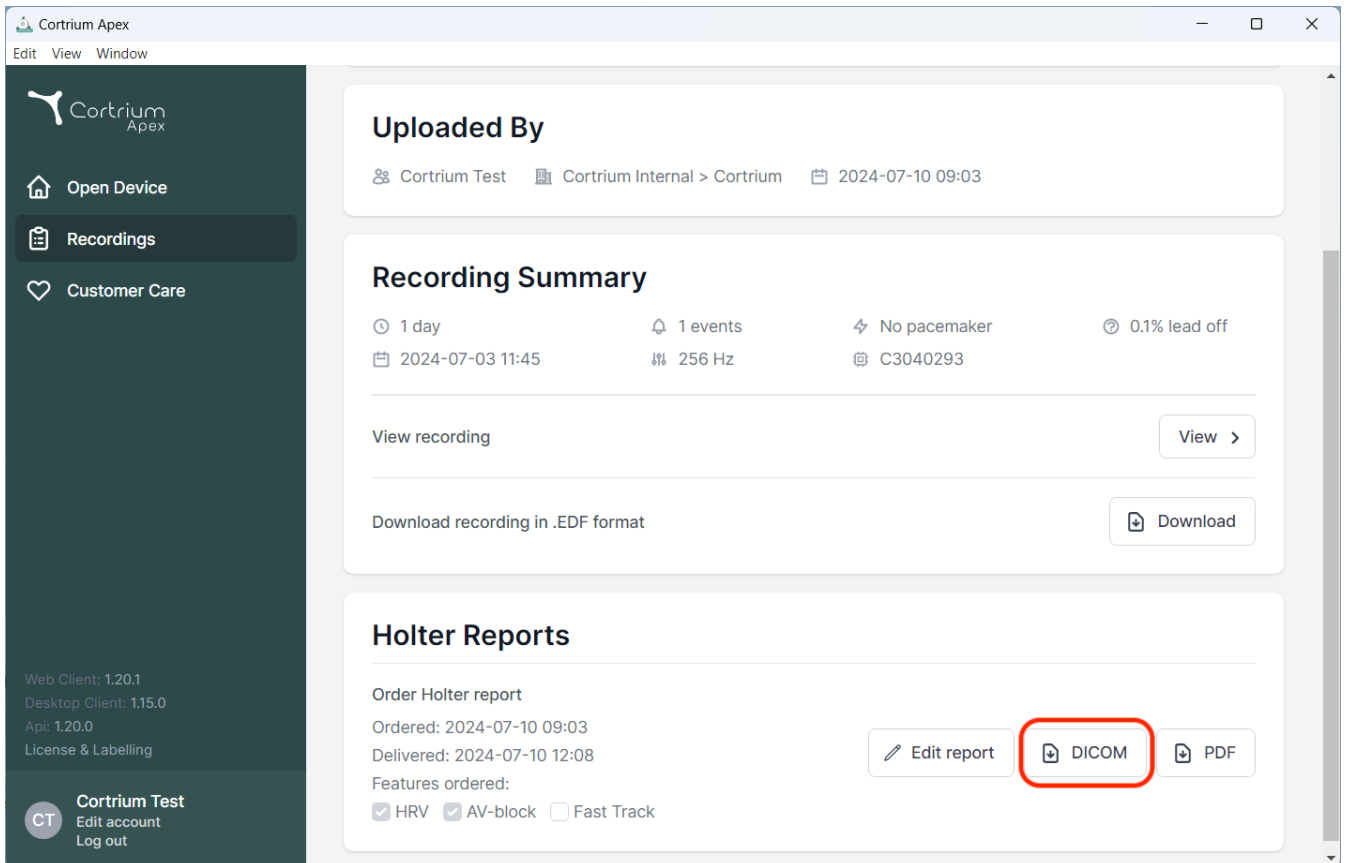The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0, License & Labelling) and a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area has a top section 'Uploaded By' showing 'Cortrium Test' and a date '2024-07-10 09:03'. Below this is the 'Recording Summary' section with details like '1 day', '1 events', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download'. The bottom section is 'Holter Reports', showing 'Order Holter report' with dates and a list of features ordered (HRV, AV-block, Fast Track). At the bottom right of this section are three buttons: 'Edit report', 'DICOM' (highlighted with a red box), and 'PDF'.

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## Account information

### Account information

If you change your email address you need to confirm the new address

Title

Initials

CT

First name

Cortrium

Last name

Test

Email address

testreports@cortrium.onmicrosoft.com

Reset password

### Account preferences

Here you change preferences for your account

### Set preferences

×

Notify when a Holter report is ready

You will receive a notification by email when a Holter report is ready.

### DICOM report preferences

Here you can change preferences for the DICOM report integration

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
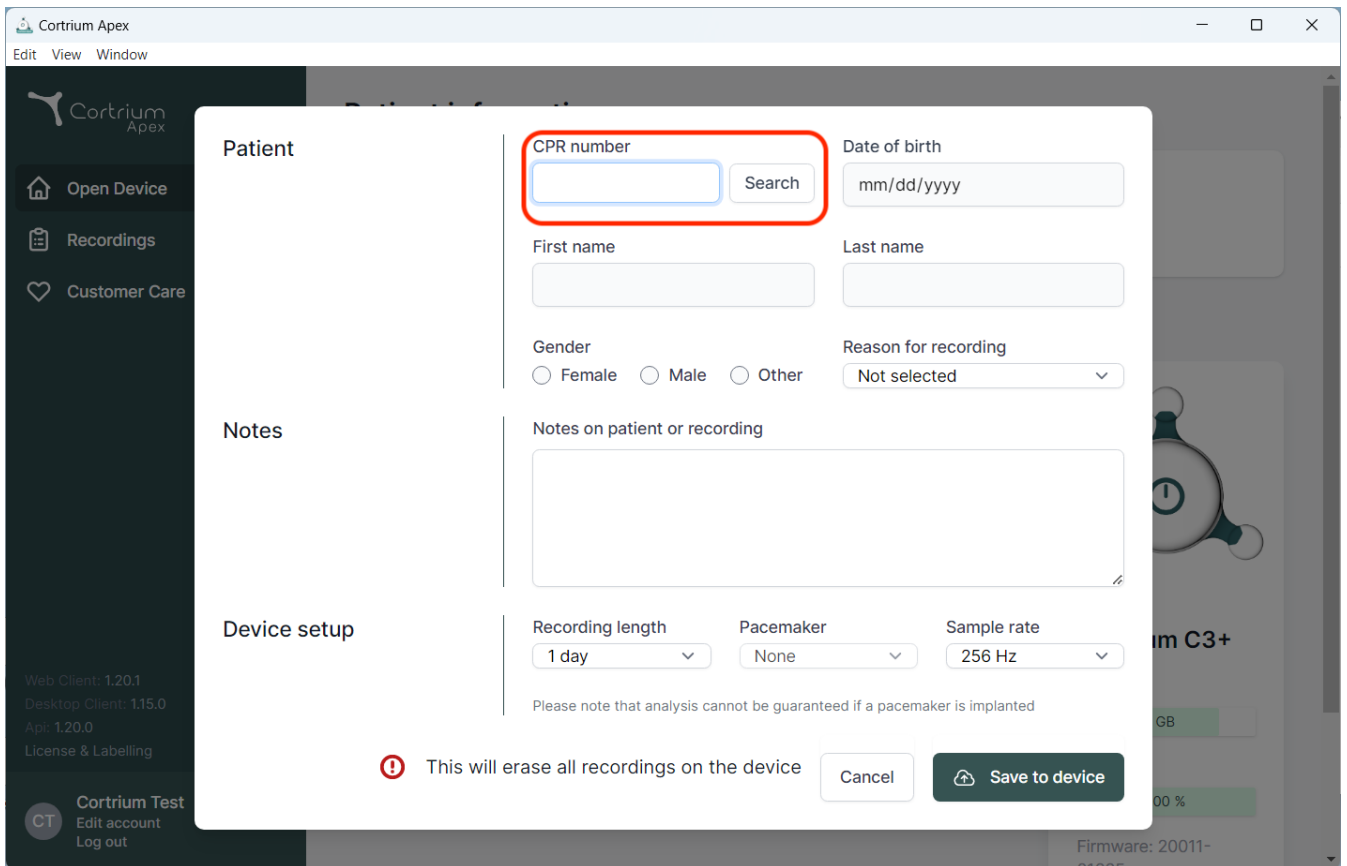
The screenshot shows the Cortrium Apex software interface with a modal window for patient search. The modal is titled "Patient" and contains the following fields and controls:

- CPR number:** A text input field with a blue border, highlighted by a red rectangle. Next to it is a "Search" button.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Three radio buttons labeled "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.
- Notes:** A section titled "Notes" with a sub-label "Notes on patient or recording" and a large text area for notes.
- Device setup:** A section with three dropdown menus: "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup section, there is a warning message: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". At the bottom of the modal, there is a red warning icon and the text "This will erase all recordings on the device", followed by "Cancel" and "Save to device" buttons.

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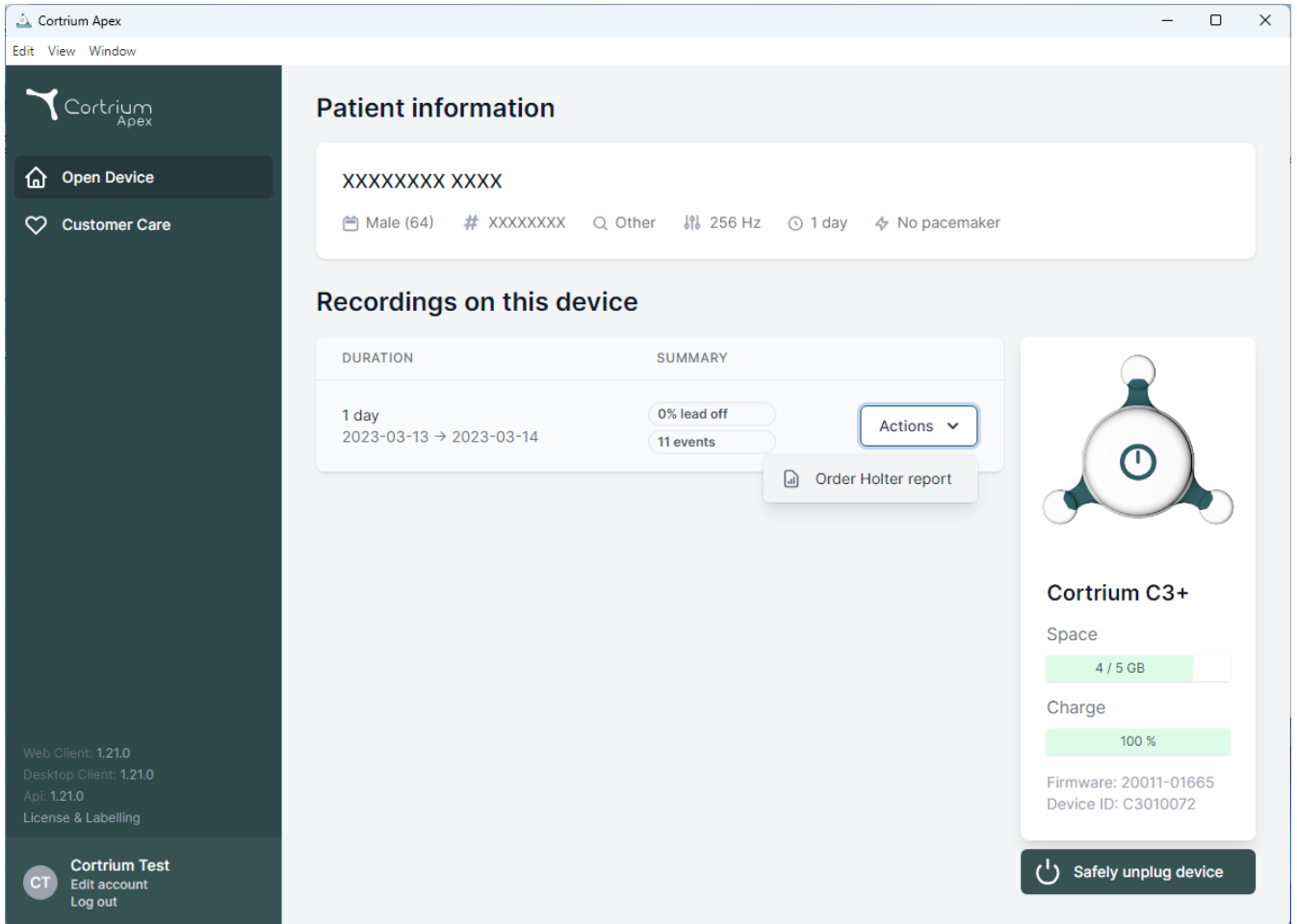The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: 'Open Device' and 'Customer Care'. The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

**Patient information**

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

**Recordings on this device**

| DURATION                         | SUMMARY                  |
|----------------------------------|--------------------------|
| 1 day<br>2023-03-13 → 2023-03-14 | 0% lead off<br>11 events |

Actions

Order Holter report

**Cortrium C3+**

Space  
4 / 5 GB

Charge  
100 %

Firmware: 20011-01665  
Device ID: C3010072

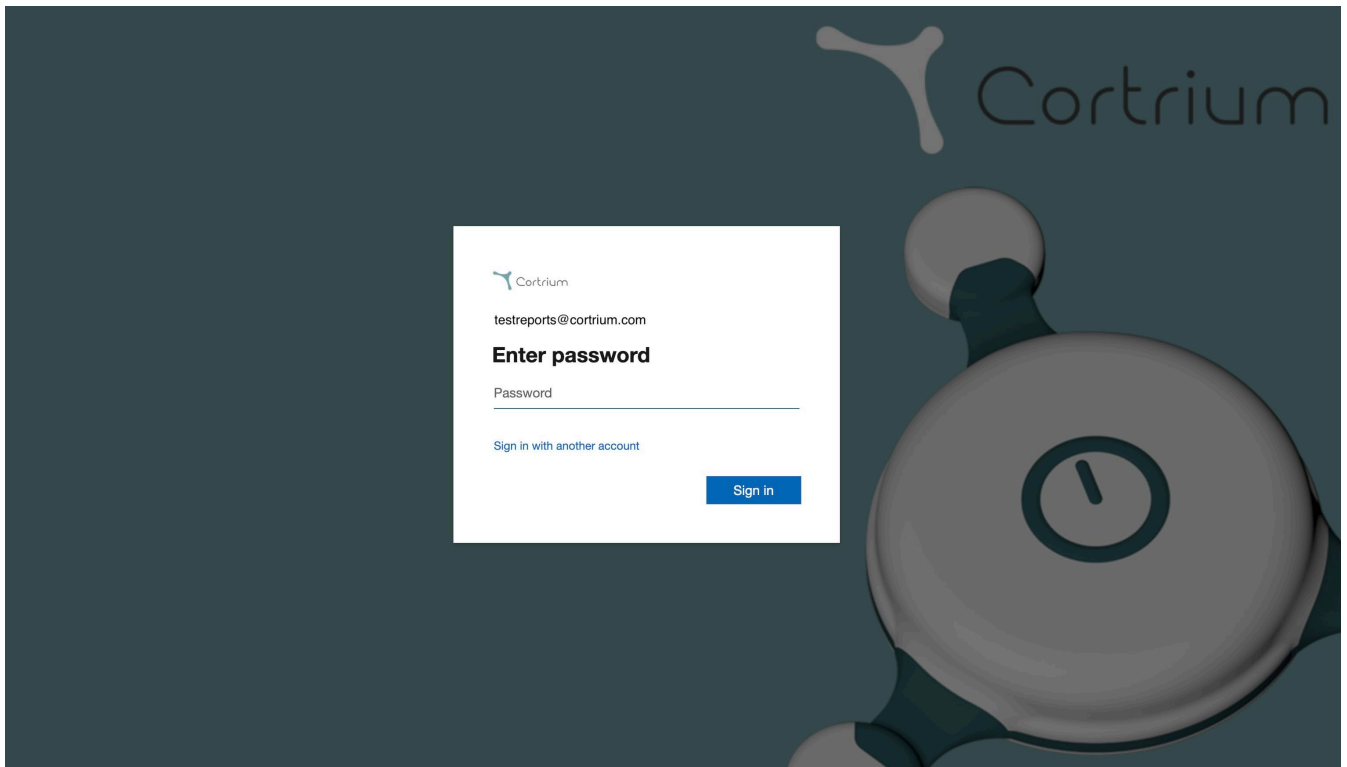
Safely unplug device

Web Client: 1.21.0  
Desktop Client: 1.21.0  
Api: 1.21.0  
License & Labelling

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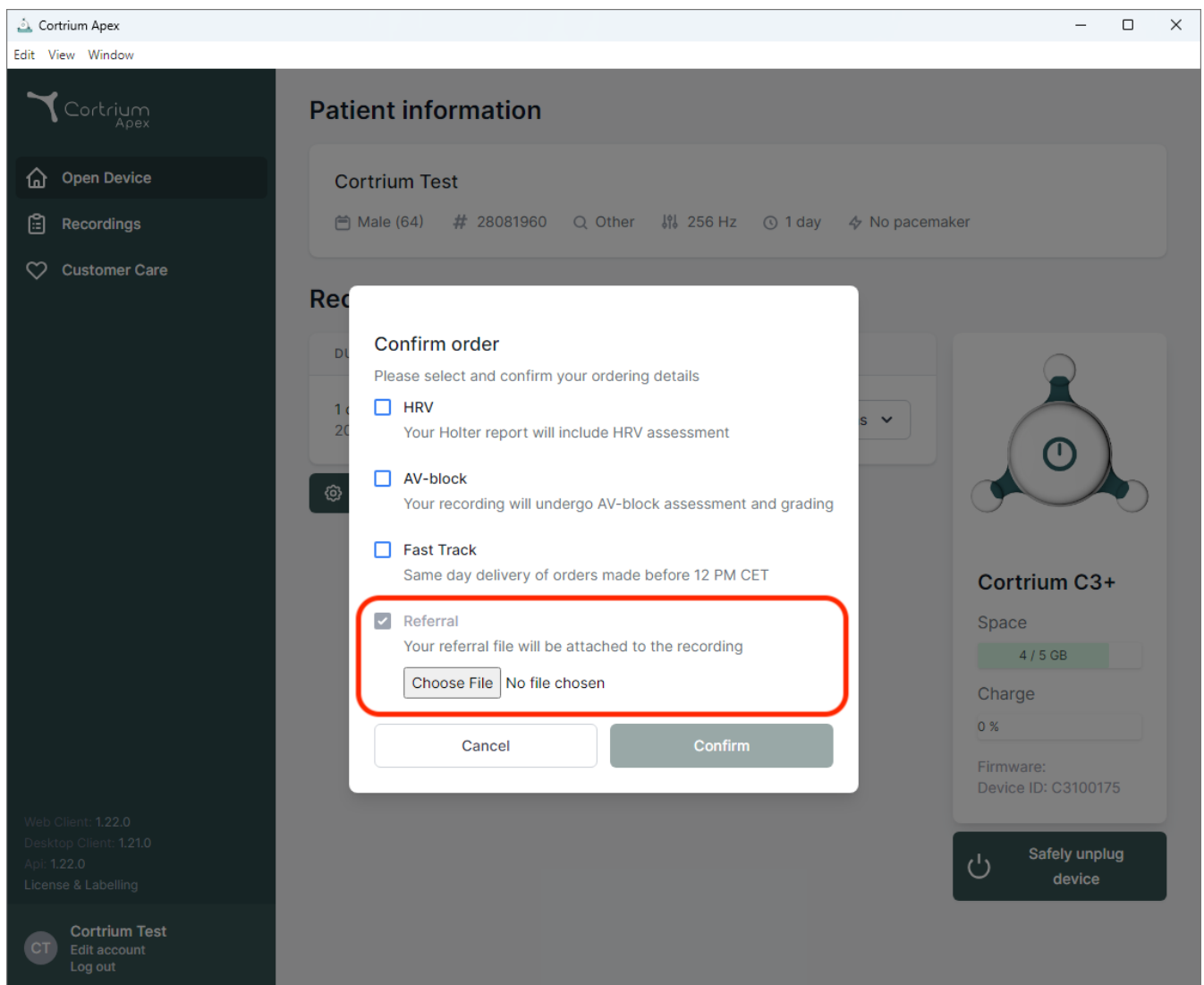
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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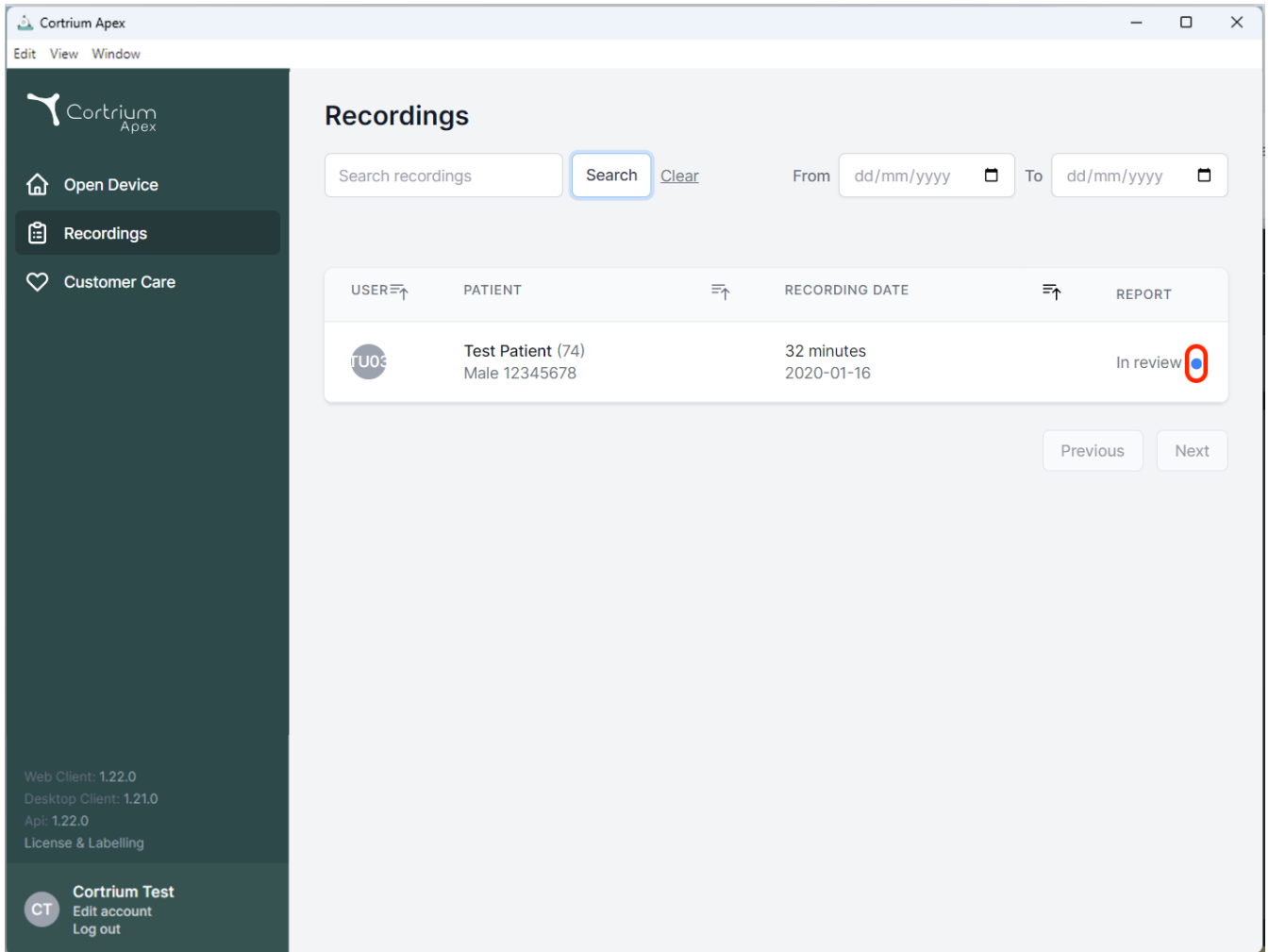
Search recordings  [Clear](#)

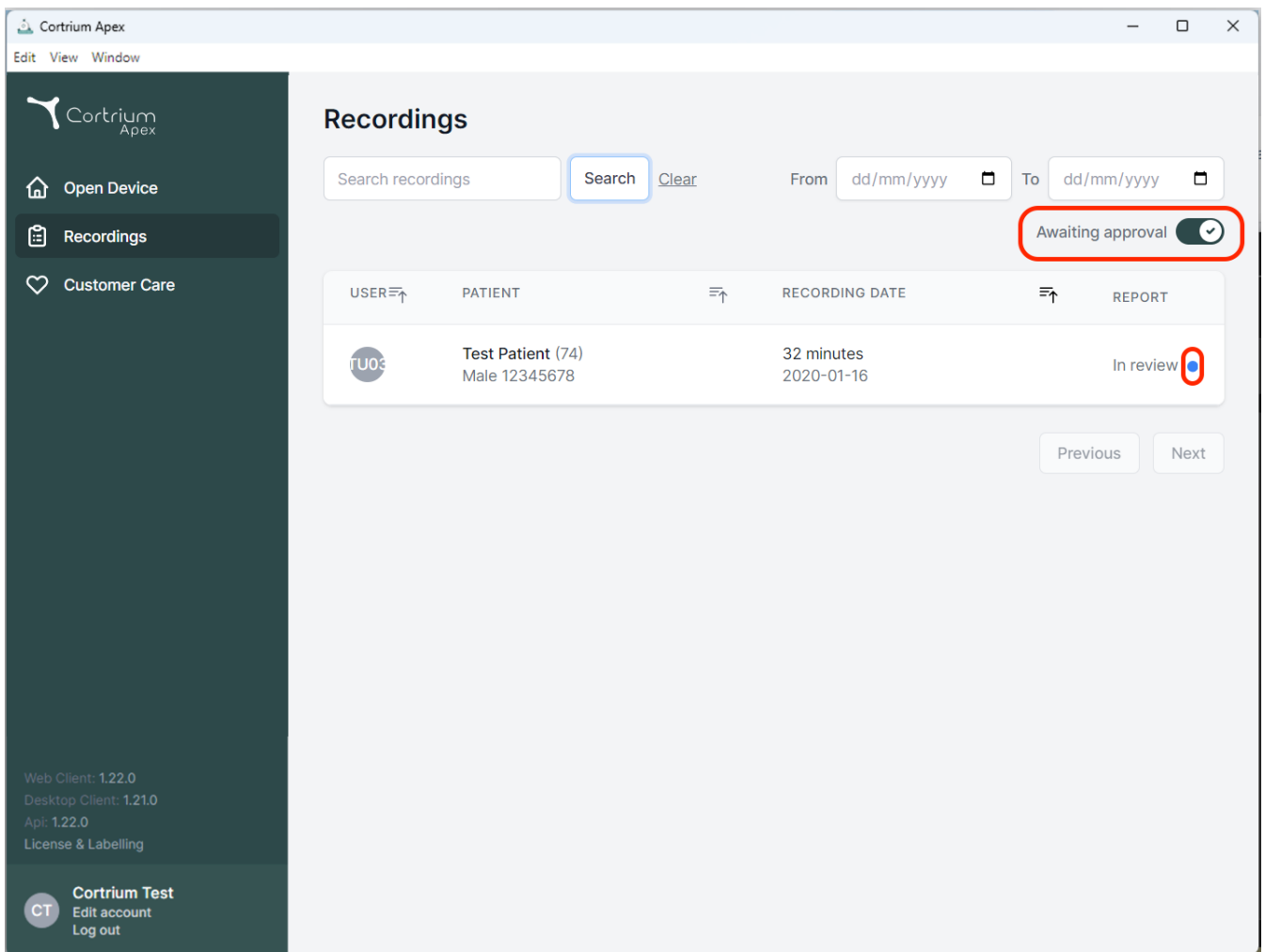
From dd/mm/yyyy To dd/mm/yyyy

| USER | PATIENT                            | RECORDING DATE           | REPORT                                                                                        |
|------|------------------------------------|--------------------------|-----------------------------------------------------------------------------------------------|
| TU03 | Test Patient (74)<br>Male 12345678 | 32 minutes<br>2020-01-16 | In review  |

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**Recordings**

Search recordings   From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒

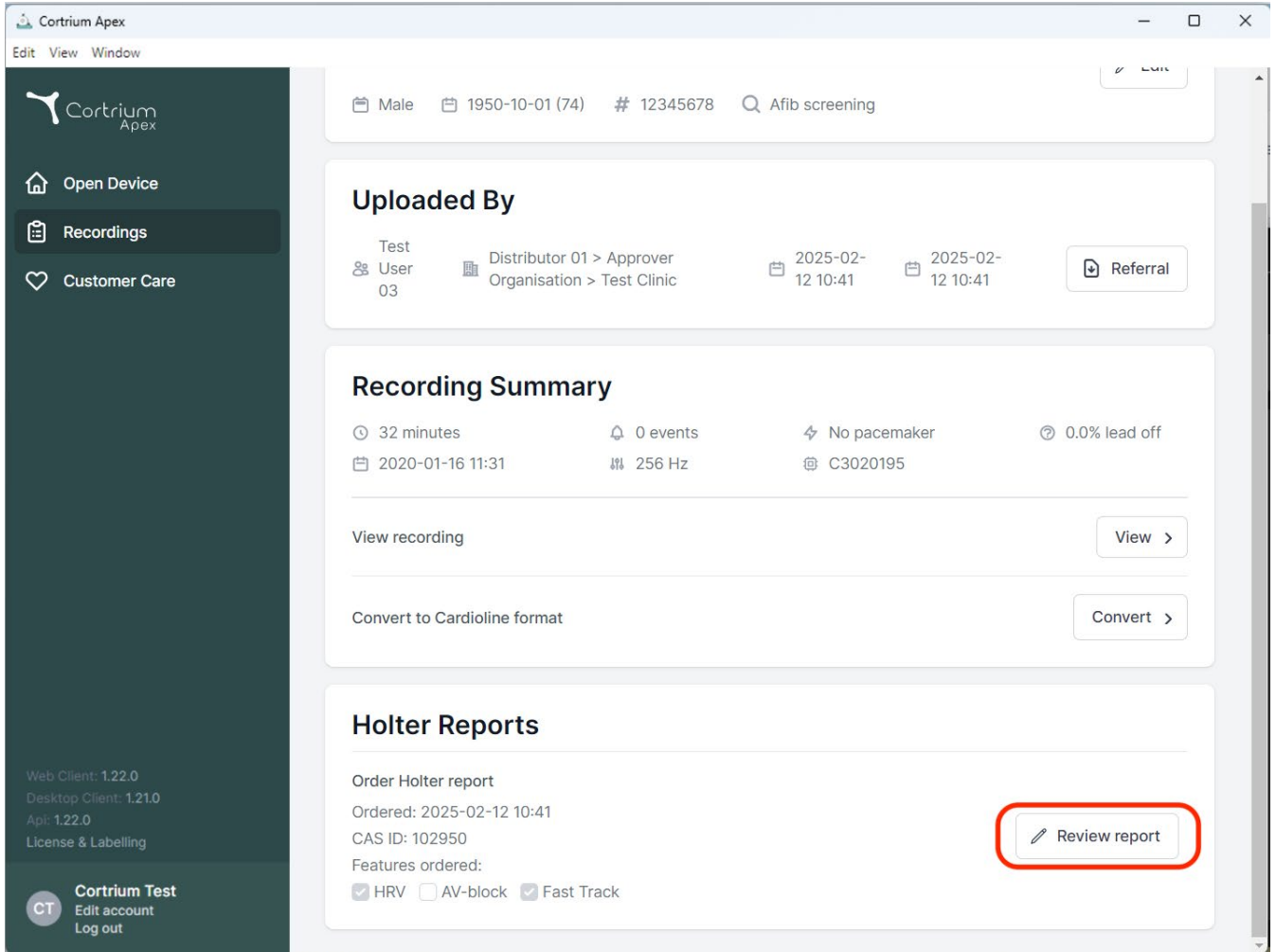
| USER | PATIENT                            | RECORDING DATE           | REPORT                                                                                          |
|------|------------------------------------|--------------------------|-------------------------------------------------------------------------------------------------|
| TU03 | Test Patient (74)<br>Male 12345678 | 32 minutes<br>2020-01-16 | In review  |

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The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient details: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section shows 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates of 2025-02-12 10:41. A 'Referral' button is present. The 'Recording Summary' section provides details: 32 minutes, 0 events, No pacemaker, 0.0% lead off, and a recording date of 2020-01-16 11:31. It includes a 'View recording' button and a 'Convert to Cardioline format' button. The 'Holter Reports' section shows 'Order Holter report' with an order date of 2025-02-12 10:41, CAS ID: 102950, and features ordered: HRV, AV-block, and Fast Track. A 'Review report' button is highlighted with a red rectangle.

**Uploaded By**

Test User 03    Distributor 01 > Approver Organisation > Test Clinic    2025-02-12 10:41    2025-02-12 10:41    [Referral](#)

**Recording Summary**

32 minutes    0 events    No pacemaker    0.0% lead off  
 2020-01-16 11:31    256 Hz    C3020195

[View recording](#)    [View >](#)

[Convert to Cardioline format](#)    [Convert >](#)

**Holter Reports**

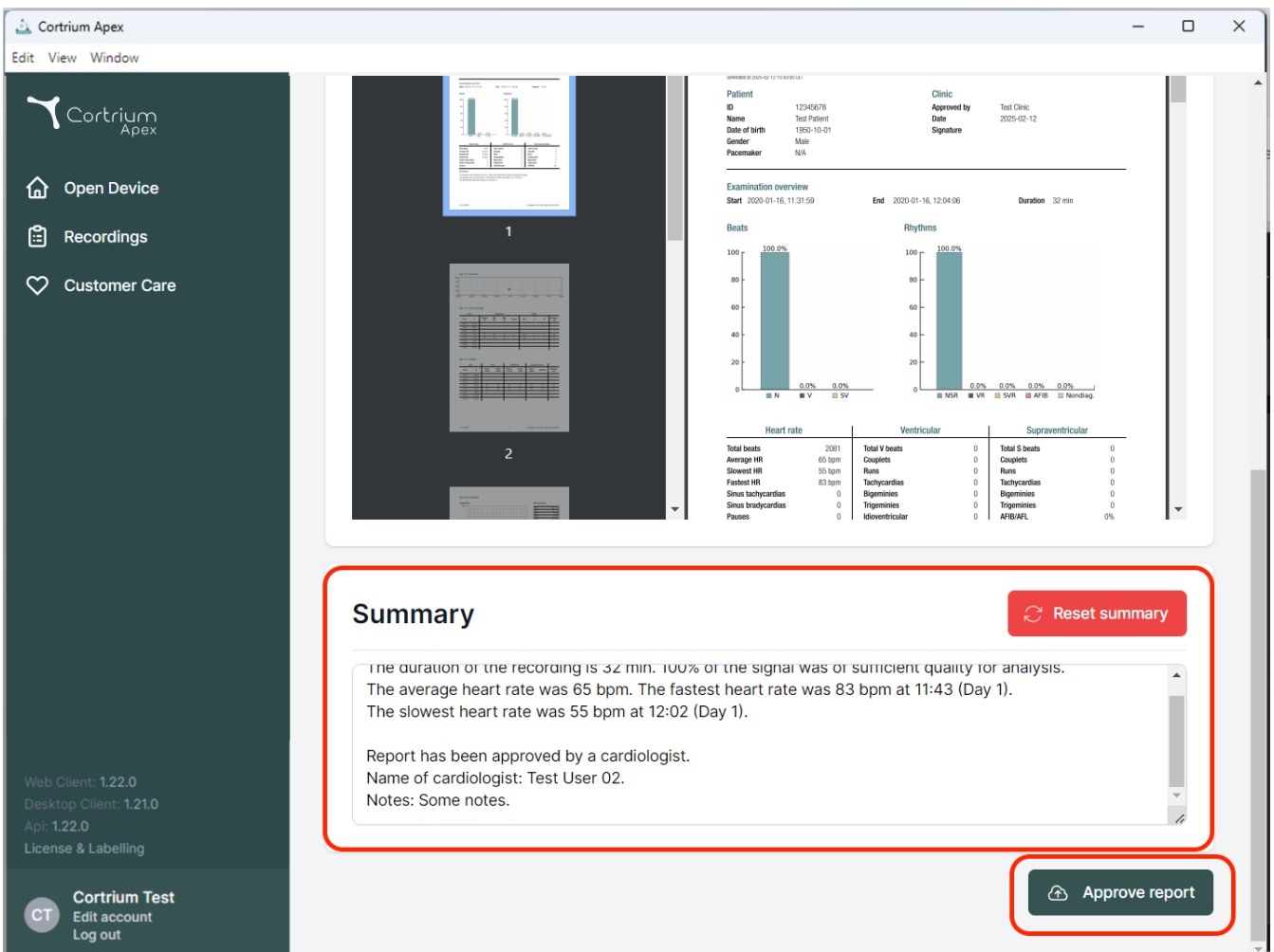
Order Holter report  
 Ordered: 2025-02-12 10:41  
 CAS ID: 102950  
 Features ordered:  
☒ HRV    ☐ AV-block    ☒ Fast Track

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**Cortrium Apex**

Edit View Window

**Open Device**  
**Recordings**  
**Customer Care**

Web Client: 1.22.0  
Desktop Client: 1.21.0  
Apl: 1.22.0  
License & Labelling

**Cortrium Test**  
Edit account  
Log out

**Patient**  
ID: 12345678  
Name: Test Patient  
Date of birth: 1980-10-01  
Gender: Male  
Pacemaker: N/A

**Clinic**  
Approved by: Test Clinic  
Date: 2025-02-12  
Signature:

**Examination overview**  
Start: 2020-01-16, 11:31:59  
End: 2020-01-16, 12:04:06  
Duration: 32 min

**Beats**  
100.0%  
0.0%  
0.0%

**Rhythms**  
100.0%  
0.0%  
0.0%  
0.0%  
0.0%

| Heart rate         |        | Ventricular     |   | Supraventricular |    |
|--------------------|--------|-----------------|---|------------------|----|
| Total beats        | 2031   | Total V beats   | 0 | Total S beats    | 0  |
| Average HR         | 65 bpm | Couplets        | 0 | Couplets         | 0  |
| Slowest HR         | 55 bpm | Runs            | 0 | Runs             | 0  |
| Fastest HR         | 83 bpm | Tachycardias    | 0 | Tachycardias     | 0  |
| Sinus tachycardias | 0      | Bigemines       | 0 | Bigemines        | 0  |
| Sinus bradycardias | 0      | Trigemines      | 0 | Trigemines       | 0  |
| Pauses             | 0      | Idioventricular | 0 | AF/IB/AFI        | 0% |

**Summary** [Reset summary](#)

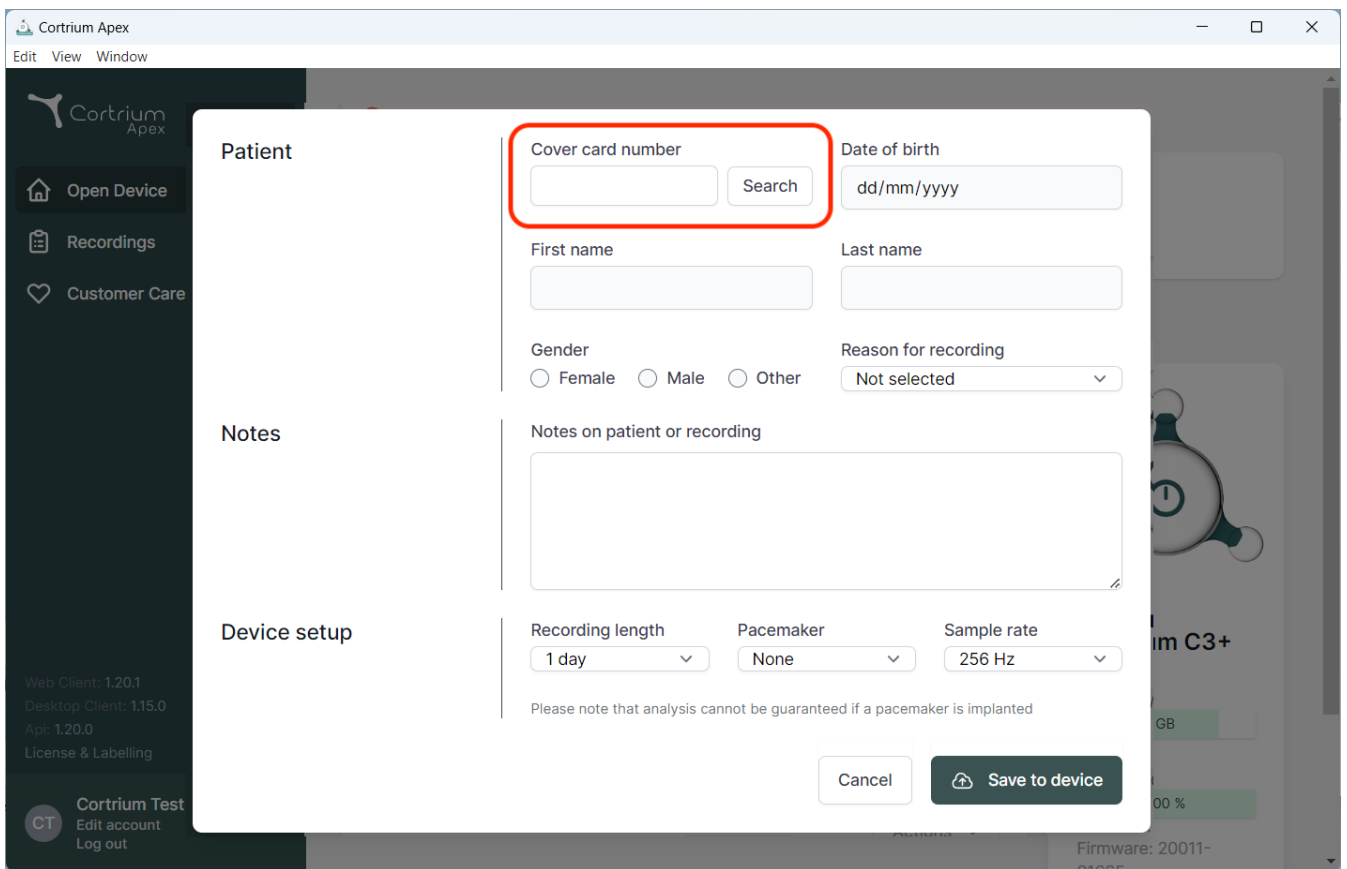
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.  
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).  
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.  
Name of cardiologist: Test User 02.  
Notes: Some notes.

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The screenshot shows the Cortrium Apex web application interface. A modal window is open for patient search and recording setup. The modal is divided into three main sections: Patient, Notes, and Device setup.

**Patient Section:**

- Cover card number:** A text input field with a "Search" button next to it. This section is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.

**Notes Section:**

- Notes on patient or recording:** A large text area for entering notes.

**Device setup Section:**

- Recording length:** A dropdown menu with "1 day" selected.
- Pacemaker:** A dropdown menu with "None" selected.
- Sample rate:** A dropdown menu with "256 Hz" selected.

Below the device setup options, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

The background of the application shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar contains version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right of the sidebar shows a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denemarken  
Email: [info@cortrium.com](mailto:info@cortrium.com) • Website: [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033

**CE**  
**0123**  
46 / 46