Cortrium Apex 1.27

Cortrium Apex 1.27

Bruksanvisning

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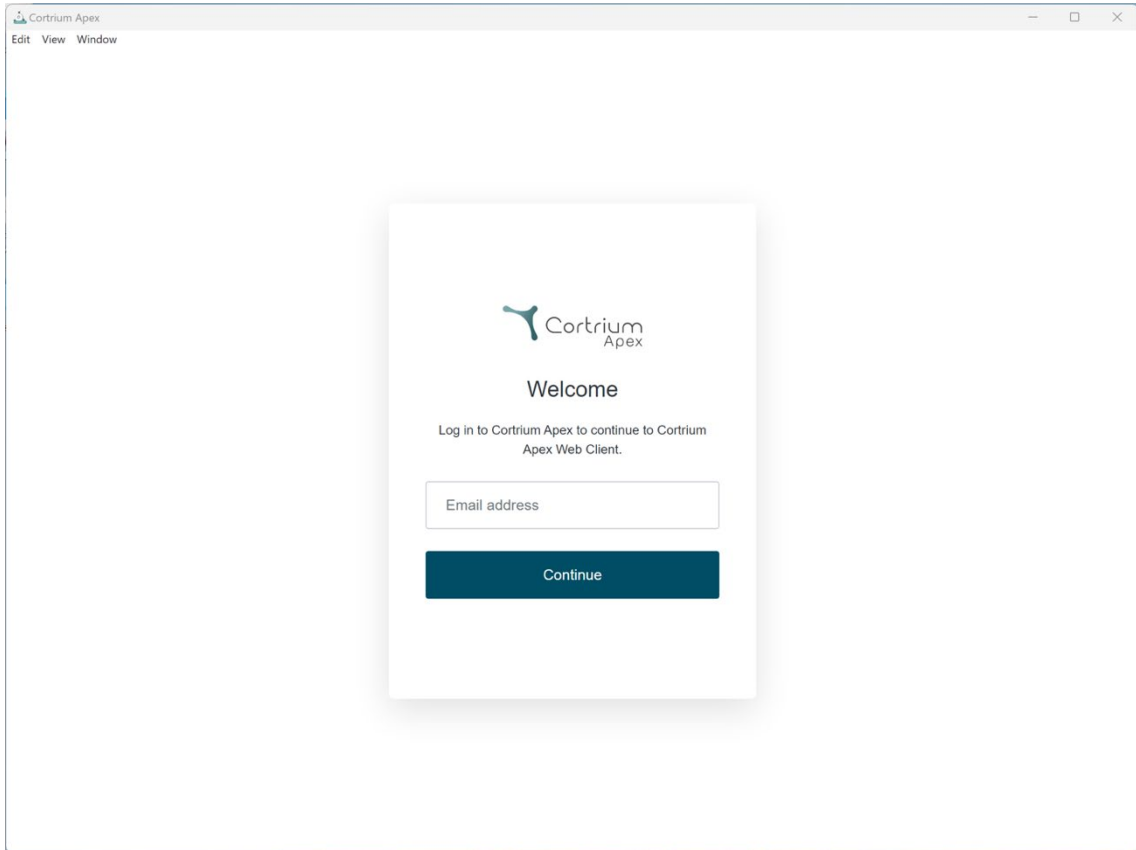
 Cortrium
Apex

Welcome

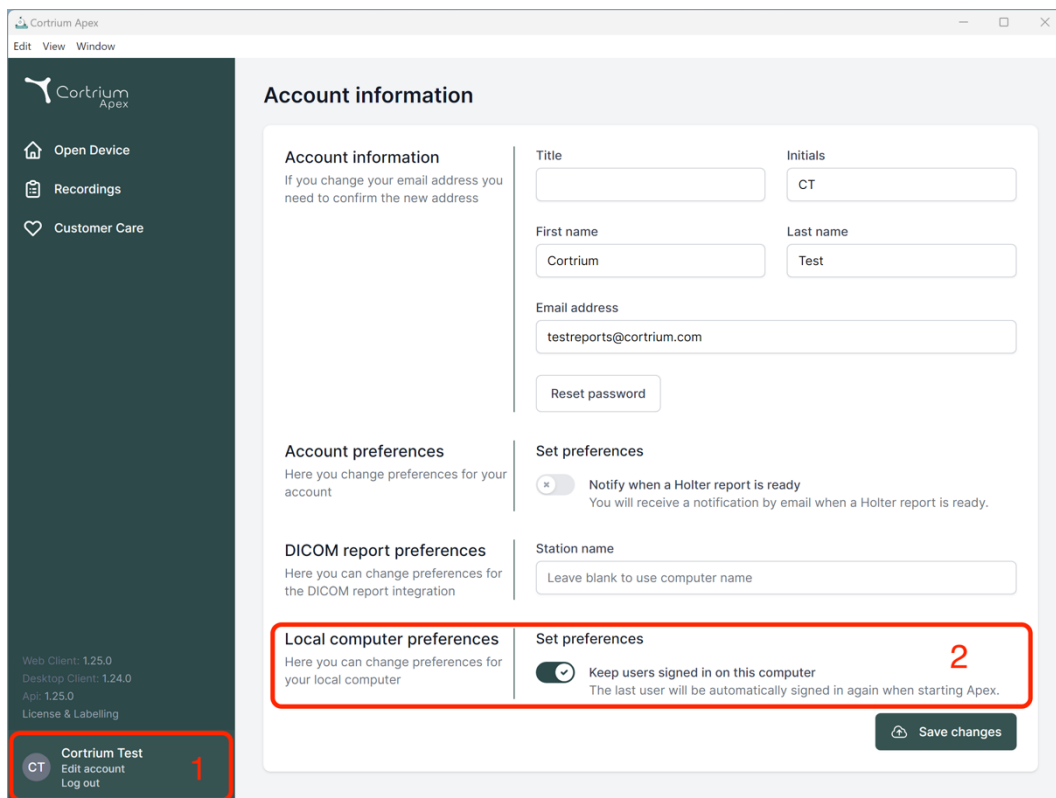
Log in to Cortrium Apex to continue to Cortrium
Apex Web Client.

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.25.0, Desktop Client: 1.24.0, Api: 1.25.0) and a user profile section for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' for changing account details; 'DICOM report preferences' for integration settings; and 'Local computer preferences' for managing local device settings. A red box labeled '1' highlights the user profile in the sidebar. Another red box labeled '2' highlights the 'Local computer preferences' section, specifically the 'Keep users signed in on this computer' toggle, which is currently turned on. A 'Save changes' button is located at the bottom right of the preferences section.

Account information

Account information
If you change your email address you need to confirm the new address

Title: Initials:

First name: Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

DICOM report preferences
Here you can change preferences for the DICOM report integration

Local computer preferences
Here you can change preferences for your local computer

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

Station name

Set preferences

☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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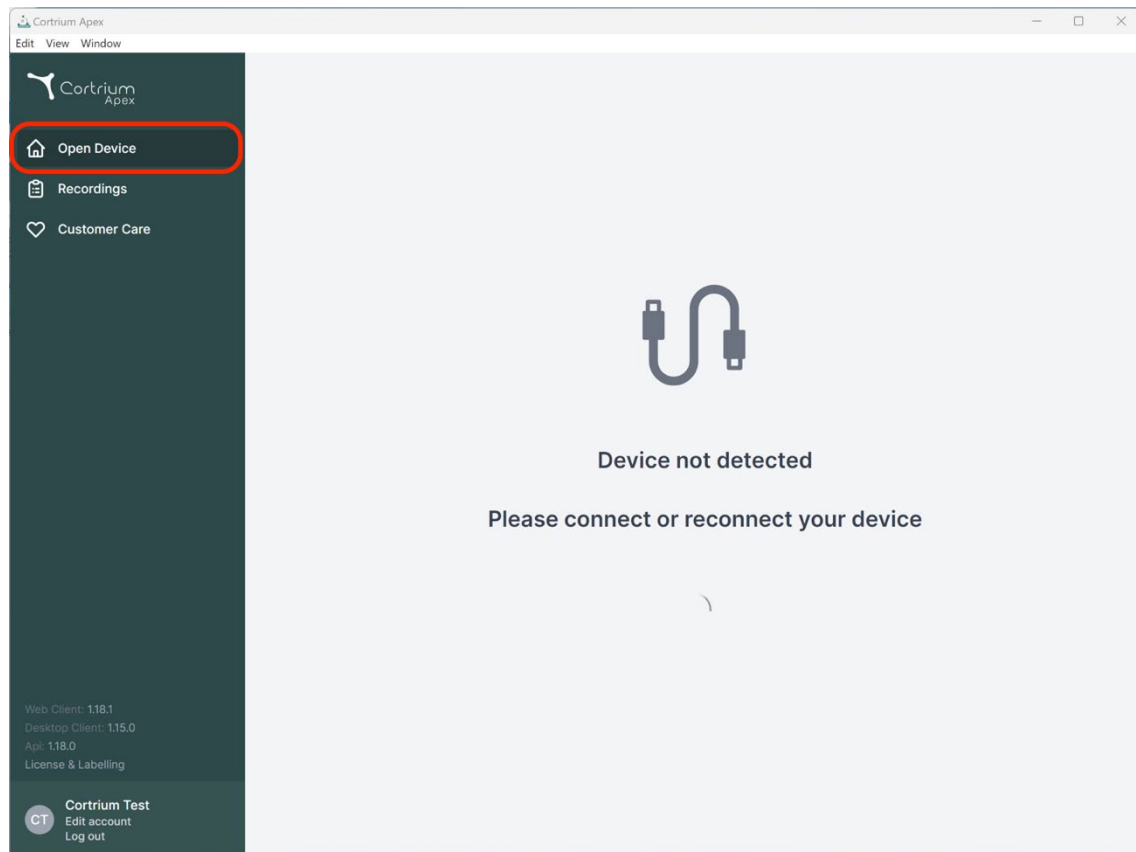
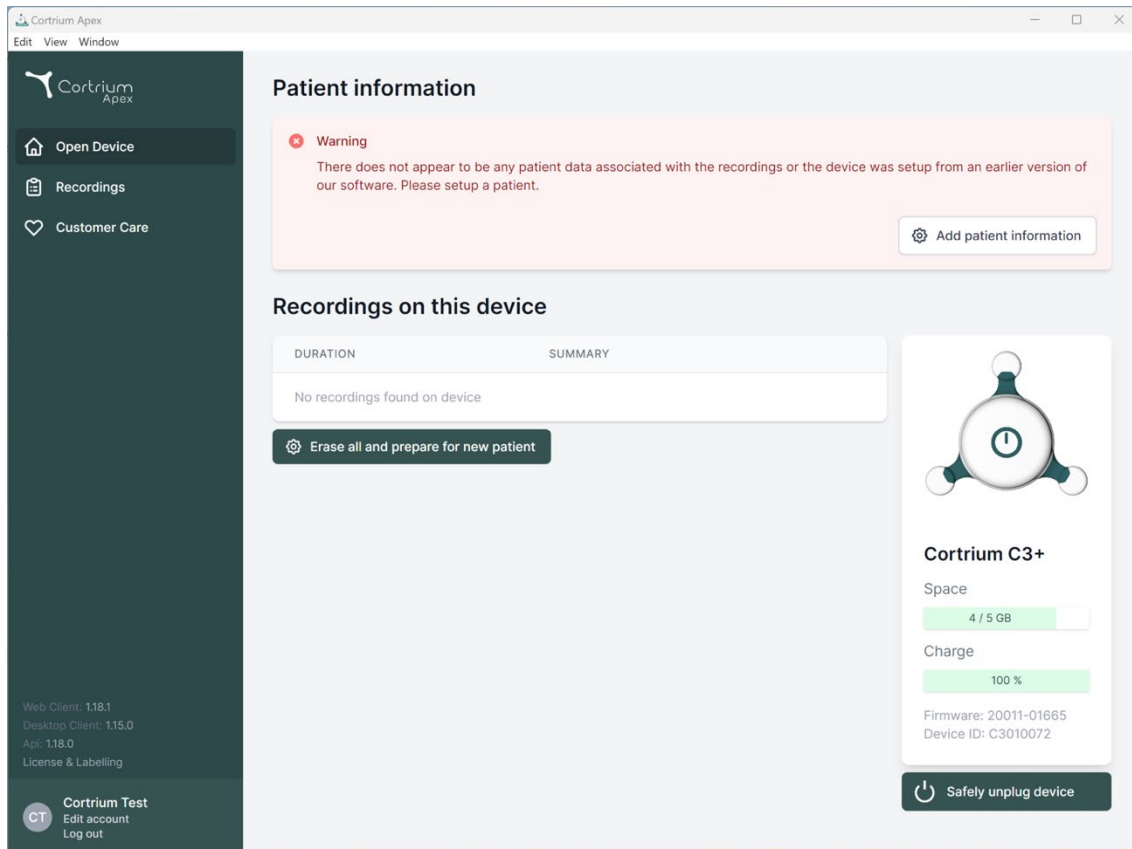
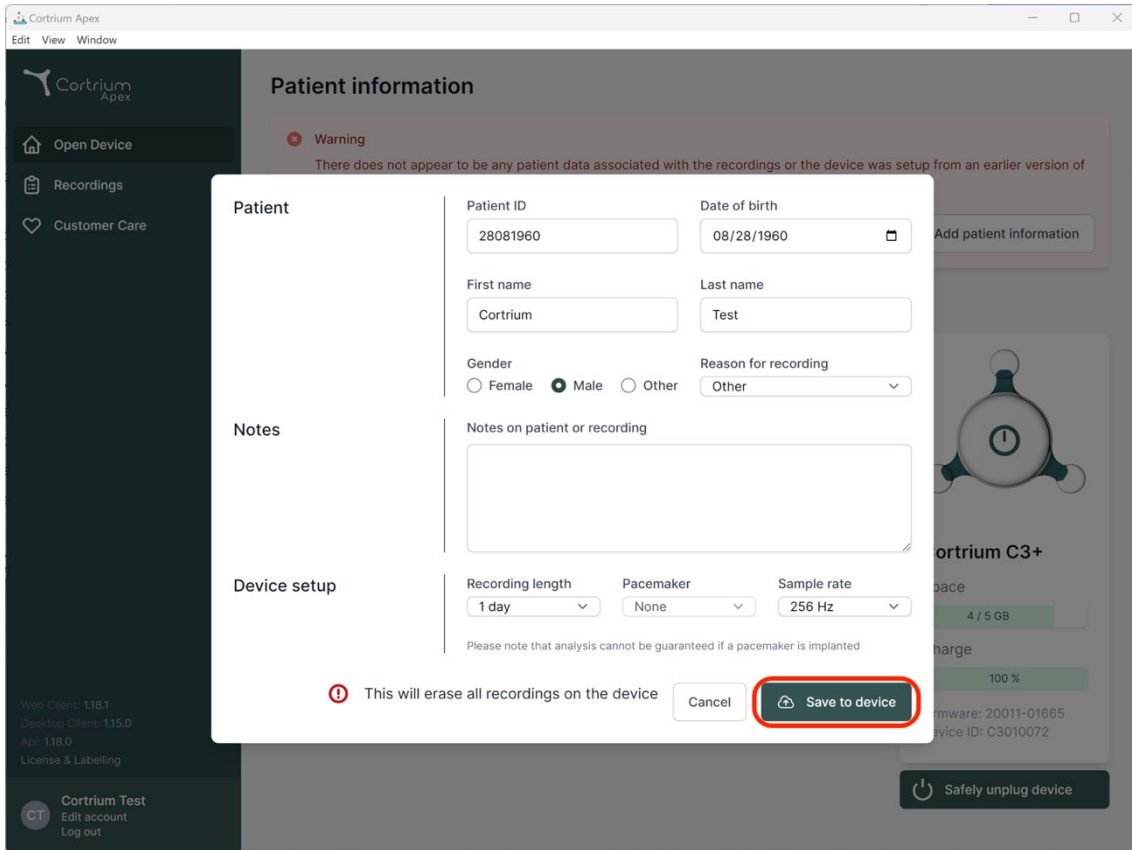
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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Buttons: Cancel, **Save to device**

Warning: This will erase all recordings on the device

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
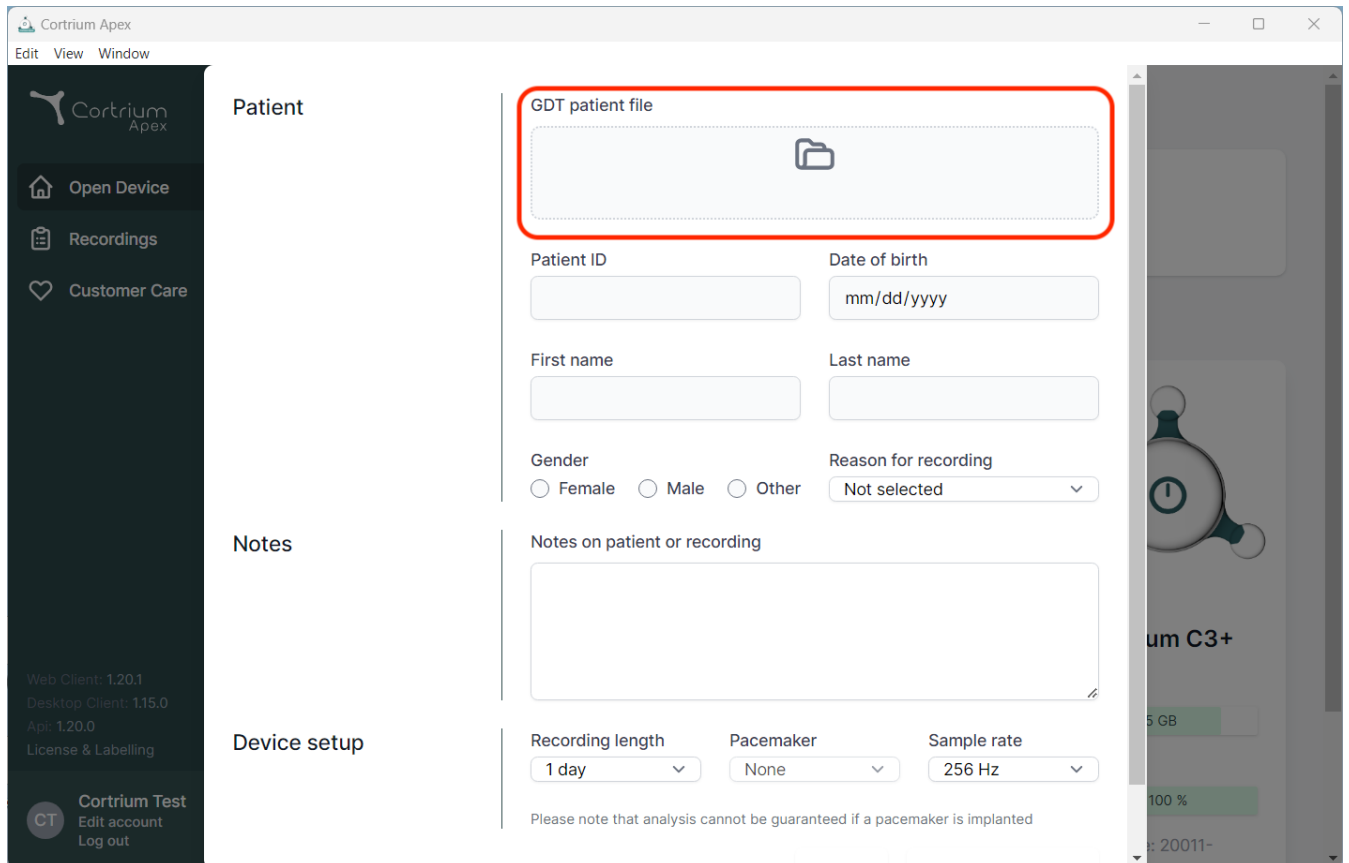
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains the Cortrium Apex logo, navigation links for 'Open Device', 'Recordings', and 'Customer Care', and version information: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', and 'Api: 1.20.0'. At the bottom of the sidebar are links for 'License & Labelling', 'Cortrium Test', 'Edit account', and 'Log out'.

The main content area is divided into three sections: 'Patient', 'Notes', and 'Device setup'. The 'Patient' section contains a 'GDT patient file' upload area (highlighted with a red box) and form fields for 'Patient ID', 'Date of birth' (mm/dd/yyyy), 'First name', 'Last name', 'Gender' (Female, Male, Other), and 'Reason for recording' (Not selected). The 'Notes' section has a text area for 'Notes on patient or recording'. The 'Device setup' section includes dropdowns for 'Recording length' (1 day), 'Pacemaker' (None), and 'Sample rate' (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'.

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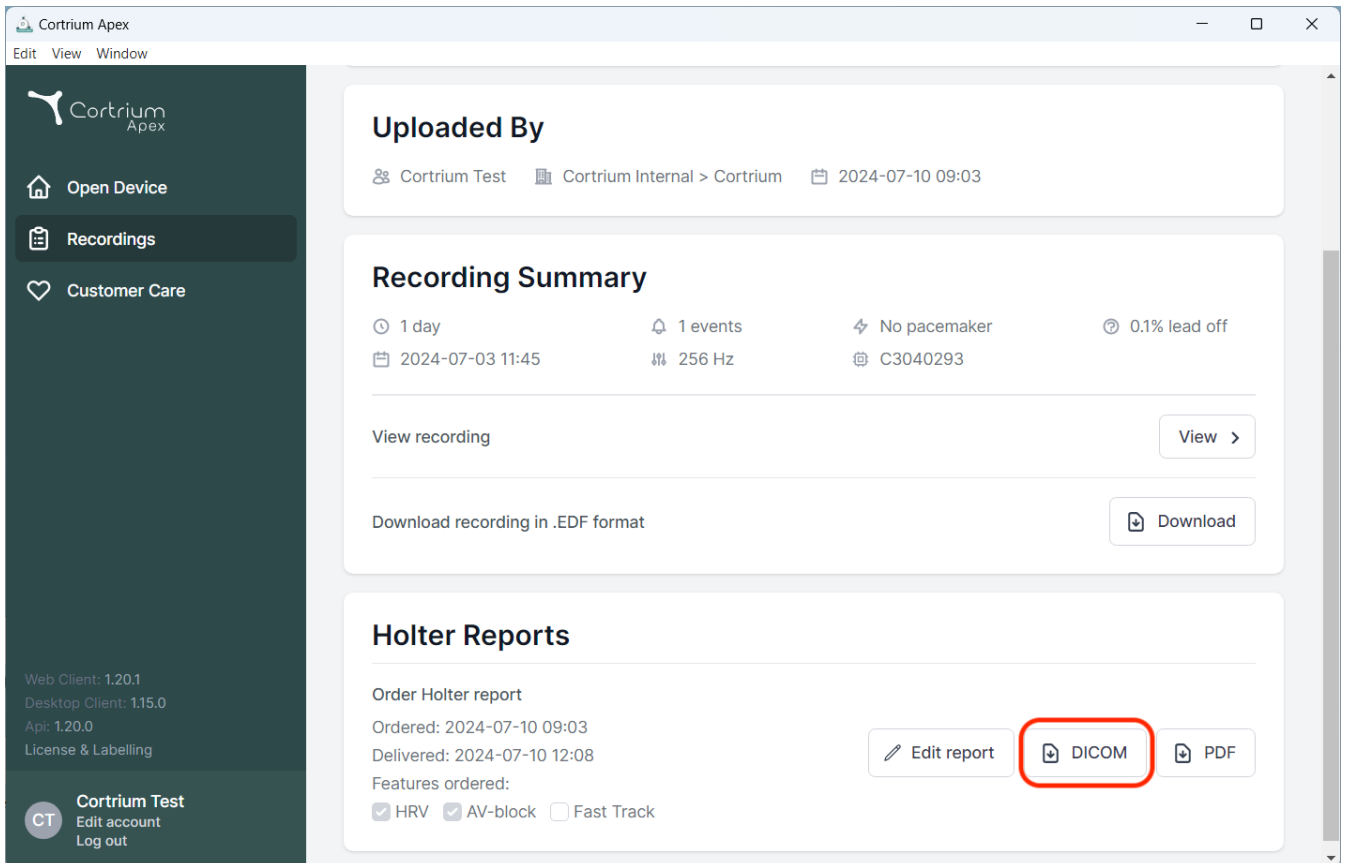
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area has a header 'Cortrium Apex' and a menu 'Edit View Window'. The 'Recordings' section displays 'Uploaded By' information: 'Cortrium Test', 'Cortrium Internal > Cortrium', and '2024-07-10 09:03'. Below this is the 'Recording Summary' section, which includes details like '1 day', '1 events', 'No pacemaker', '0.1% lead off', and '2024-07-03 11:45'. There are buttons for 'View recording' and 'Download recording in .EDF format'. The 'Holter Reports' section shows 'Order Holter report' details, including 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. At the bottom of this section, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'. The 'DICOM' button has a download icon. Below the buttons, there are checkboxes for 'HRV', 'AV-block', and 'Fast Track'.

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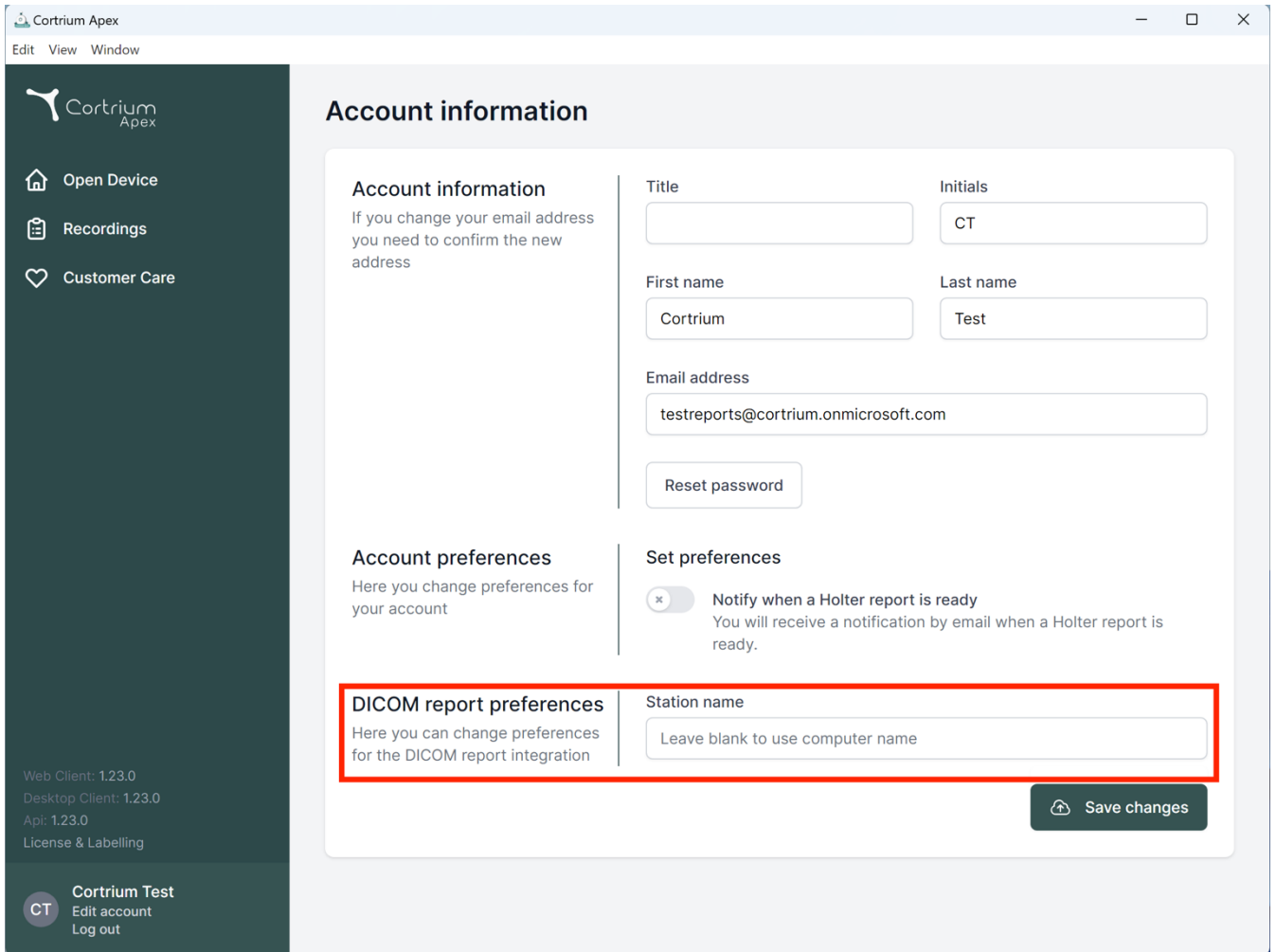The screenshot displays the 'Cortrium Apex' web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.23.0, Desktop Client: 1.23.0, Api: 1.23.0, License & Labelling) and a user profile section for 'Cortrium Test' with 'Edit account' and 'Log out' options. The main content area is titled 'Account information' and contains several sections: 'Account information' with a note about email changes, fields for Title, First name (Cortrium), Last name (Test), and Email address (testreports@cortrium.onmicrosoft.com), and a 'Reset password' button; 'Account preferences' with a note about changing preferences; 'Set preferences' with a toggle for 'Notify when a Holter report is ready'; and 'DICOM report preferences' with a note about changing preferences for DICOM integration. The 'DICOM report preferences' section is highlighted with a red border and contains a 'Station name' field with the placeholder text 'Leave blank to use computer name'. A 'Save changes' button is located at the bottom right of the main content area.

Account information

If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

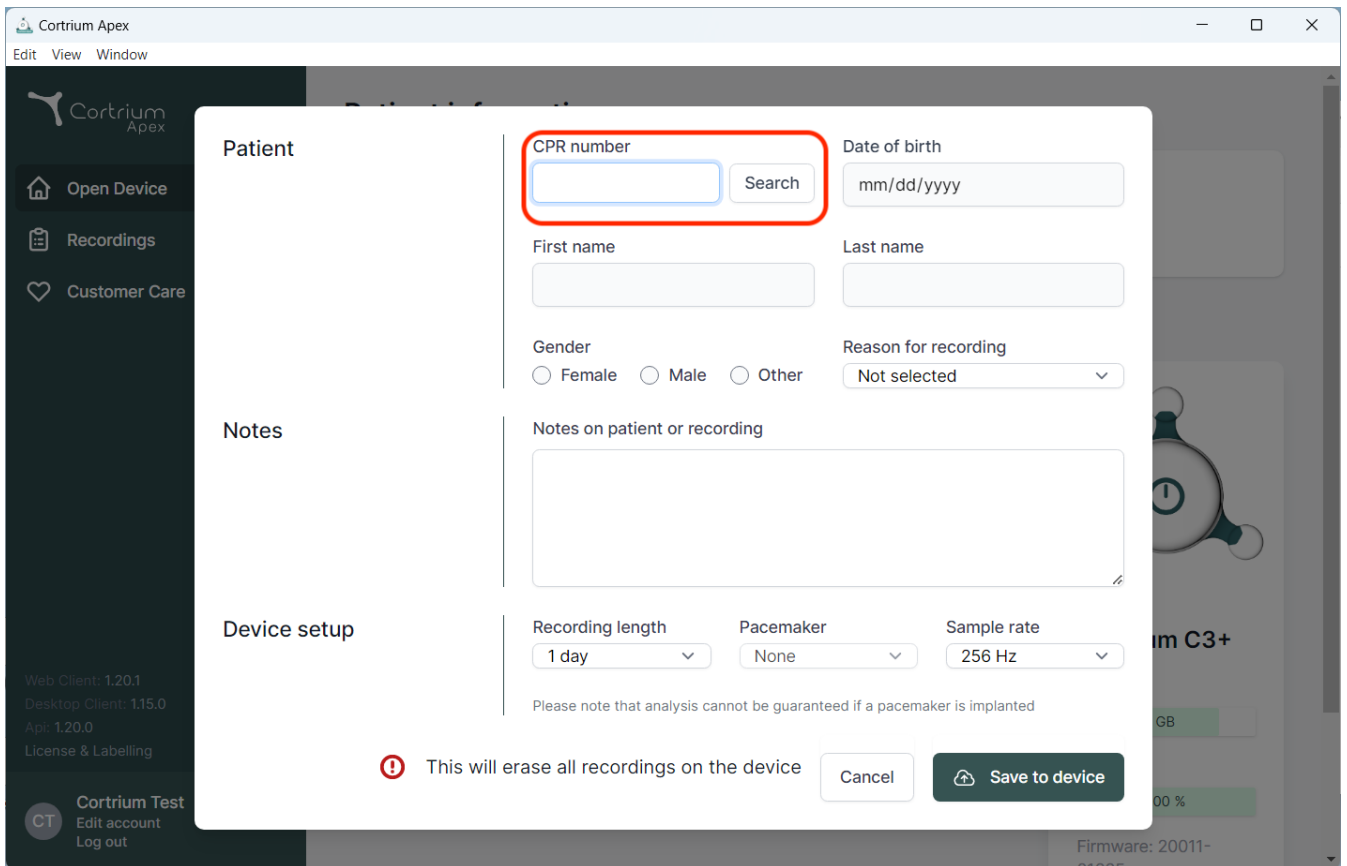
Here you can change preferences for the DICOM report integration

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
The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying a form for entering patient information. The form is divided into several sections:

- Patient:** Contains fields for "CPR number" (highlighted with a red box), "Date of birth" (mm/dd/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), and "Reason for recording" (Not selected).
- Notes:** A large text area for "Notes on patient or recording".
- Device setup:** Includes "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz).

At the bottom of the modal, there is a warning message: "This will erase all recordings on the device" with a red exclamation mark icon. Below this message are two buttons: "Cancel" and "Save to device".

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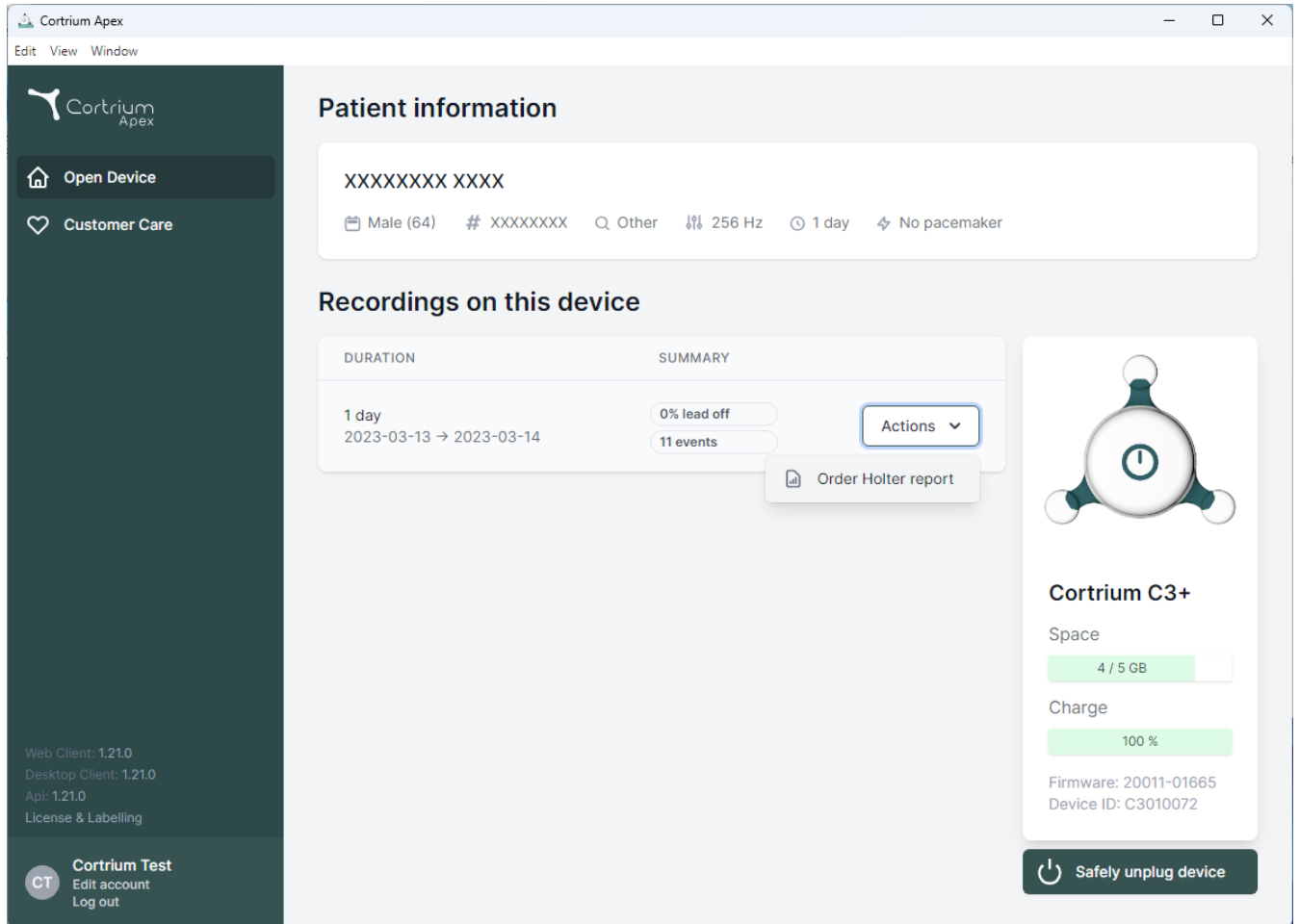The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device" and "Customer Care". The main content area is divided into two sections: "Patient information" and "Recordings on this device".

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Order Holter report

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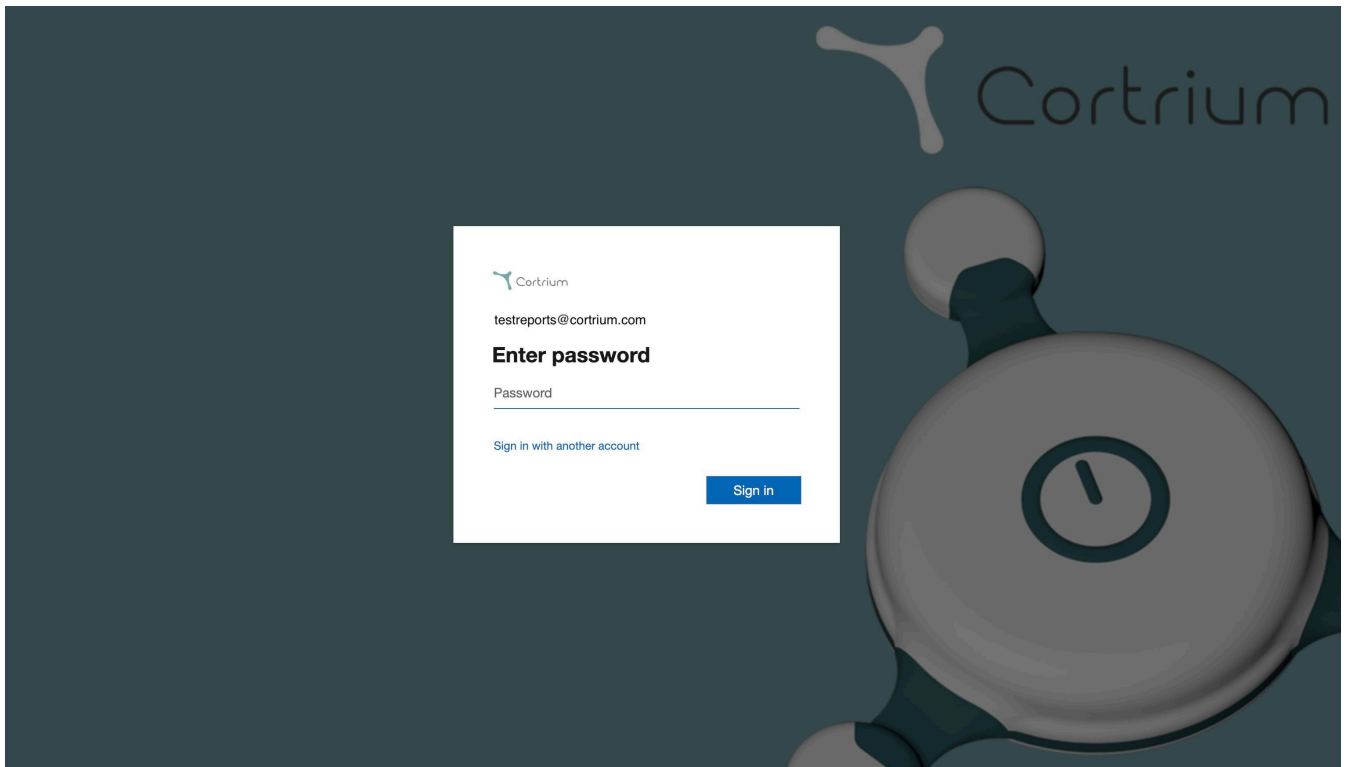
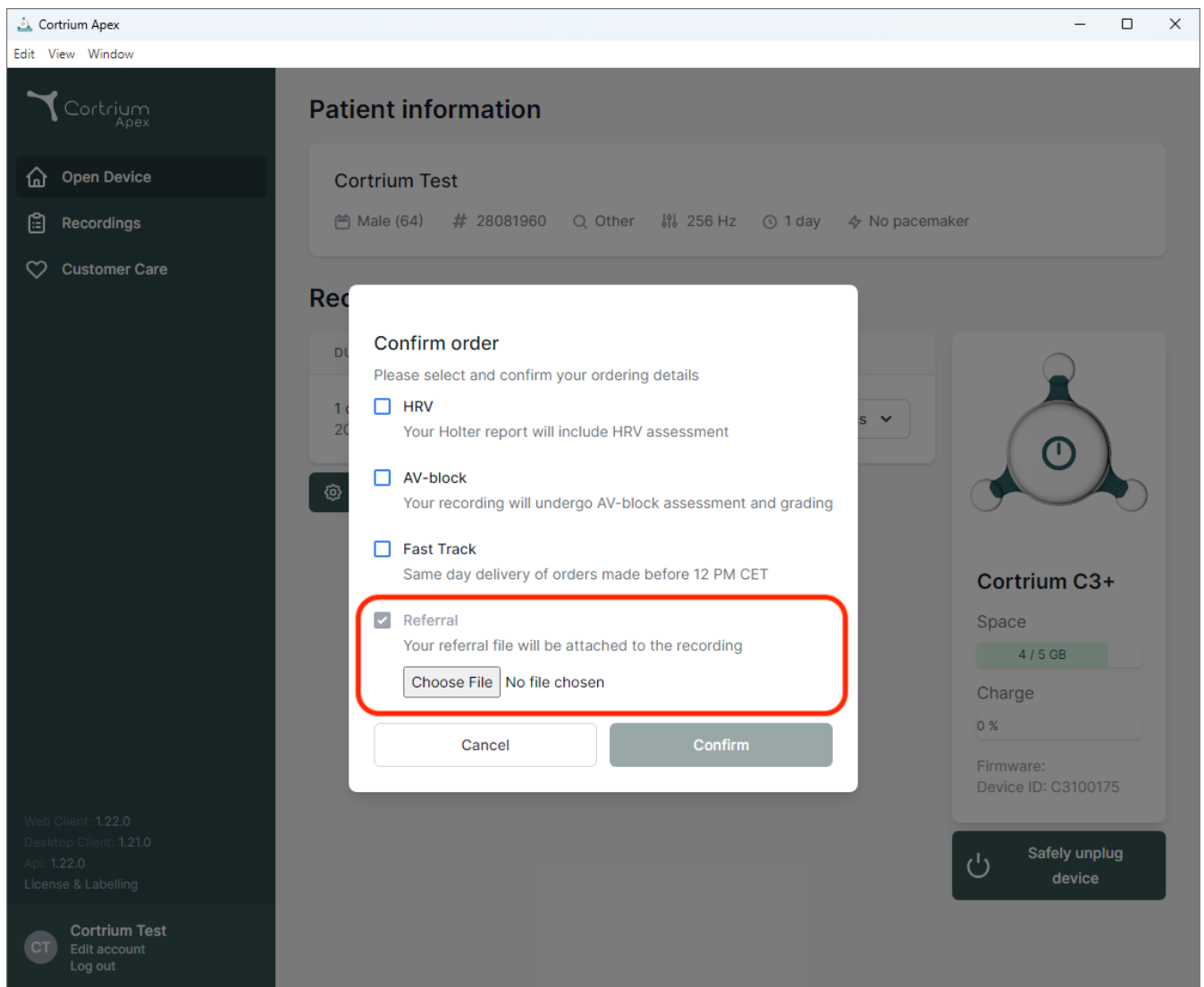
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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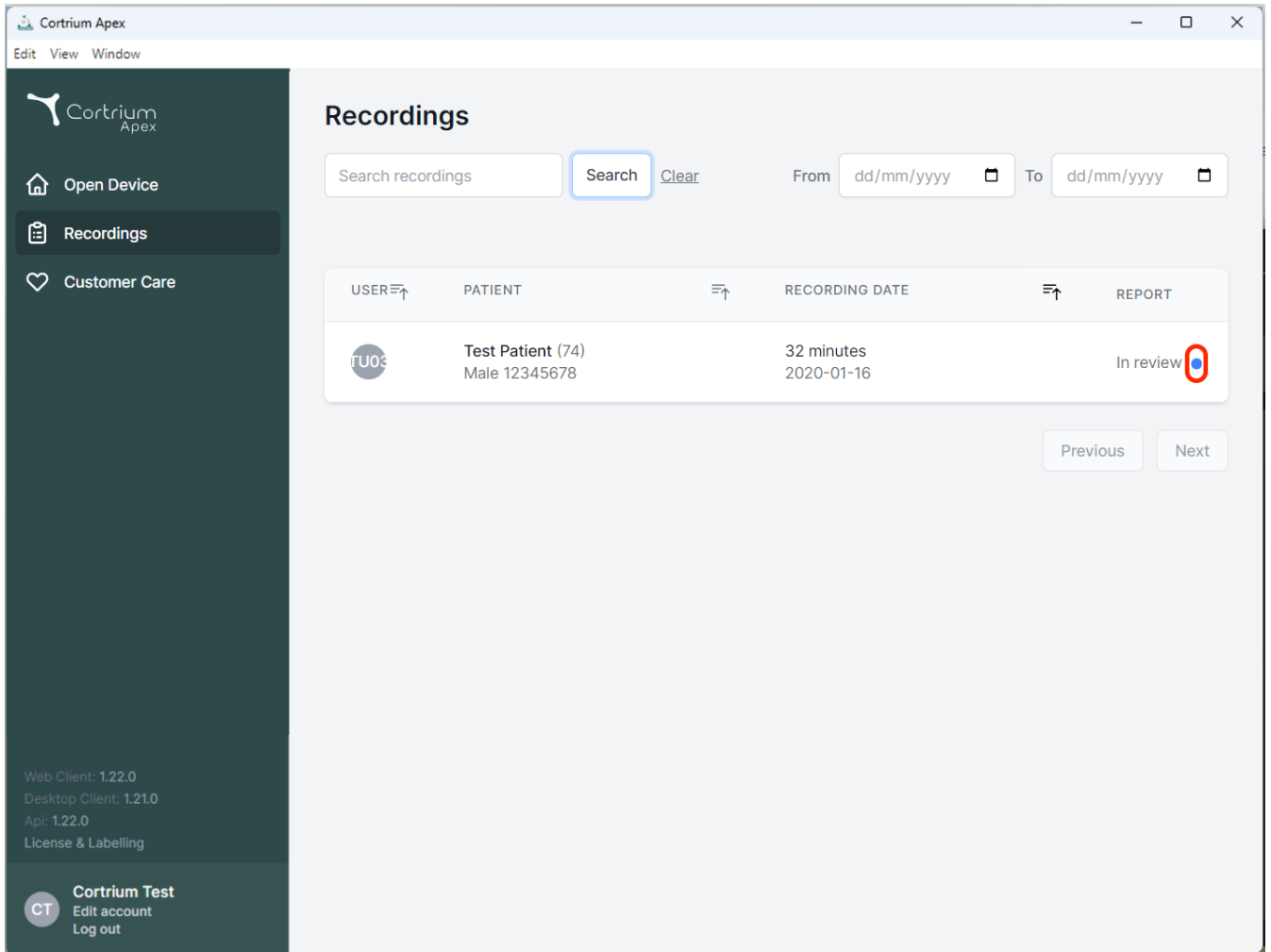
Search recordings Search Clear


From dd/mm/yyyy To dd/mm/yyyy

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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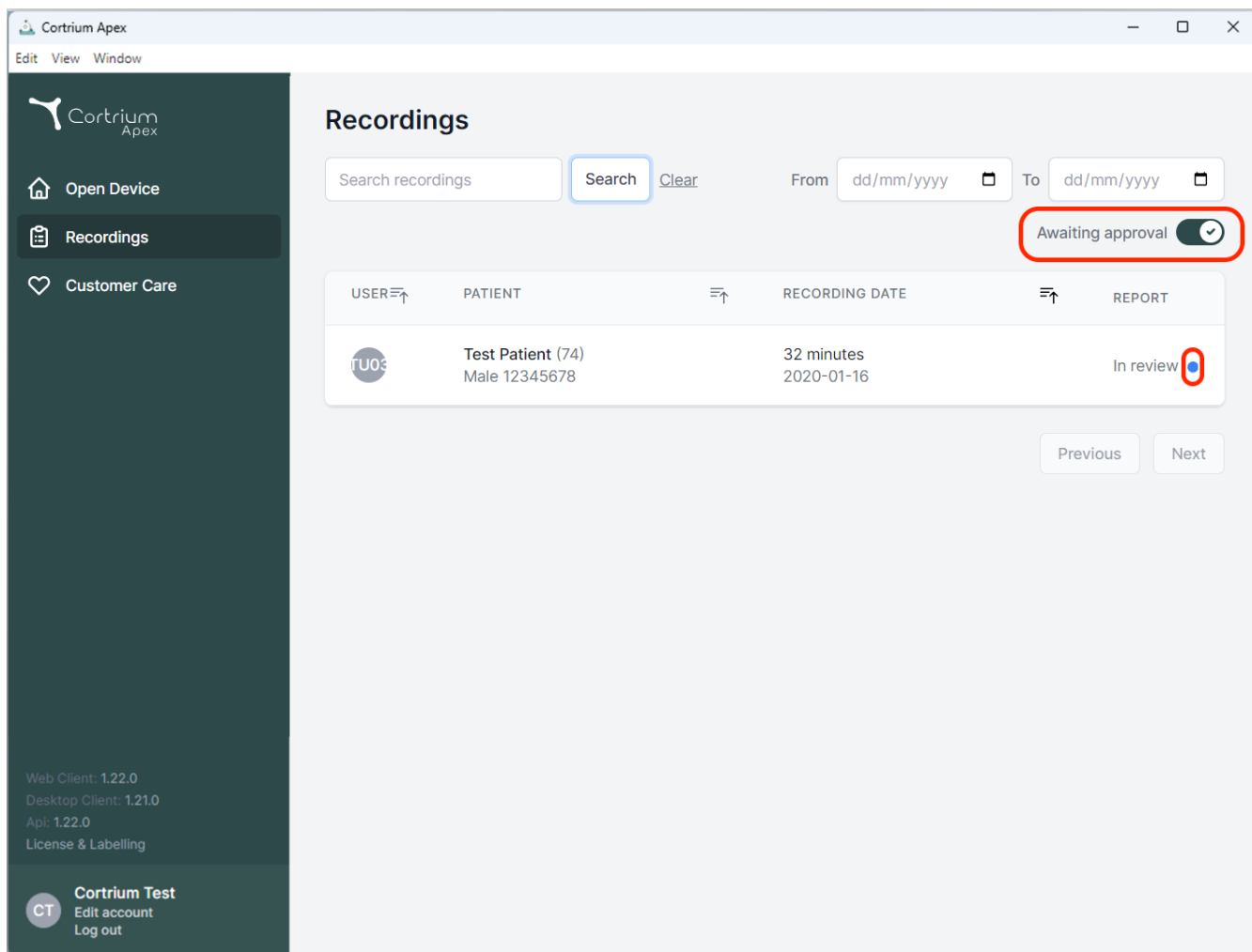Recordings

Search recordings **Search** [Clear](#) From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
U03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review 

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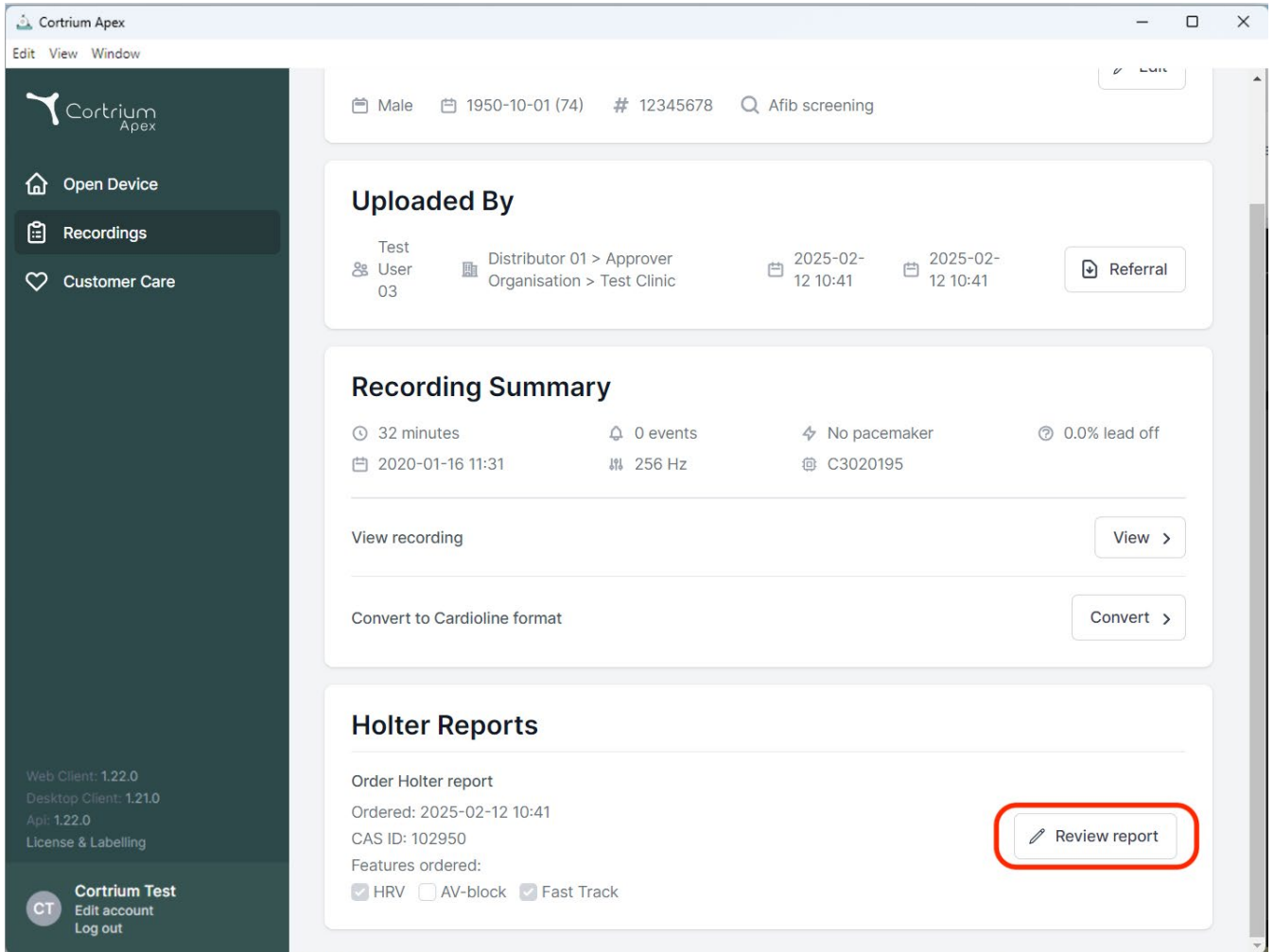
The screenshot shows the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area displays patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section shows 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with upload dates of 2025-02-12 10:41. A 'Referral' button is present. The 'Recording Summary' section lists: 32 minutes, 0 events, No pacemaker, 0.0% lead off, 2020-01-16 11:31, 256 Hz, and C3020195. It includes 'View recording' and 'Convert to Cardioline format' buttons. The 'Holter Reports' section shows 'Order Holter report' with 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

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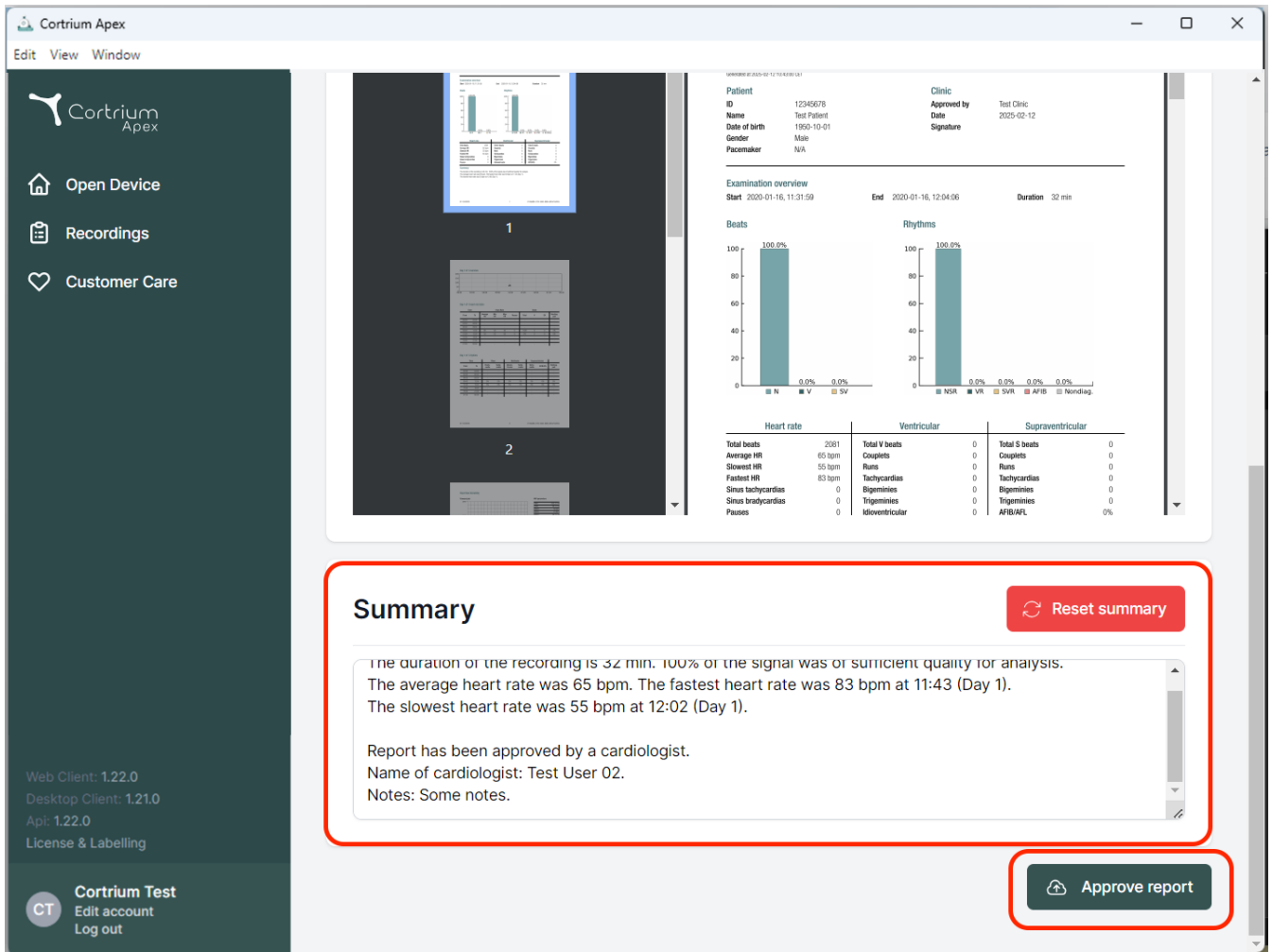The screenshot shows the Cortrium Apex web interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: Open Device, Recordings, and Customer Care. The main area displays a patient report for 'Test Patient' (ID: 12345678, Date of birth: 1950-10-01, Gender: Male, Pacemaker: N/A). The report includes an examination overview (Start: 2020-01-16, 11:31:59; End: 2020-01-16, 12:04:06; Duration: 32 min) and two bar charts: 'Beats' (100.0% for N, 0.0% for V, 0.0% for SV) and 'Rhythms' (100.0% for NSR, 0.0% for VR, 0.0% for SVR, 0.0% for AFB, 0.0% for Nonrhythm). Below the charts are three tables: Heart rate, Ventricular, and Supraventricular. The 'Summary' section is highlighted with a red border and contains the following text:

Summary [Reset summary](#)

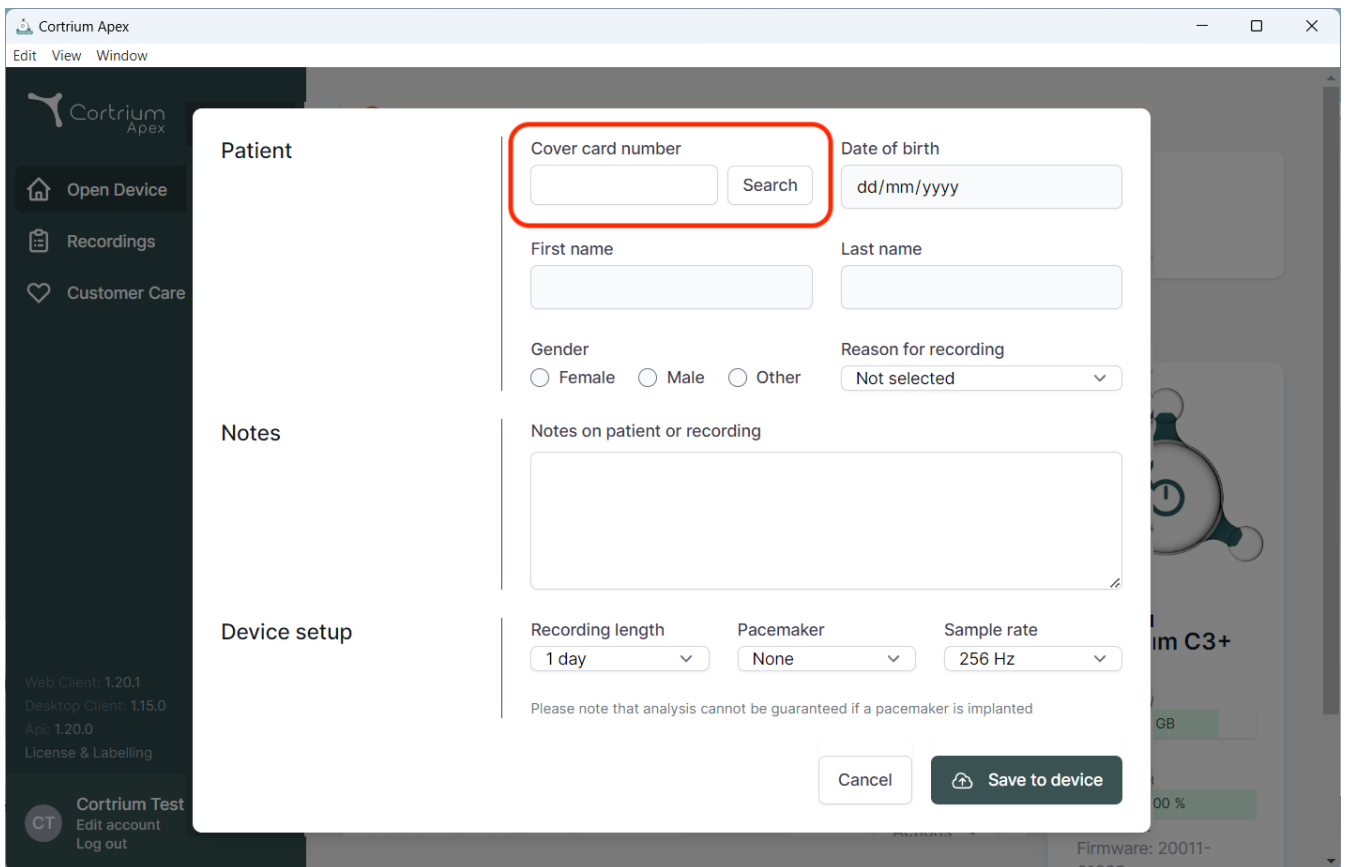
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
 Name of cardiologist: Test User 02.
 Notes: Some notes.

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Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Cancel Save to device

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark
E-post: info@cortrium.com • Nettsted: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
46 / 46