Cortrium Apex 1.27

Cortrium Apex 1.27

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
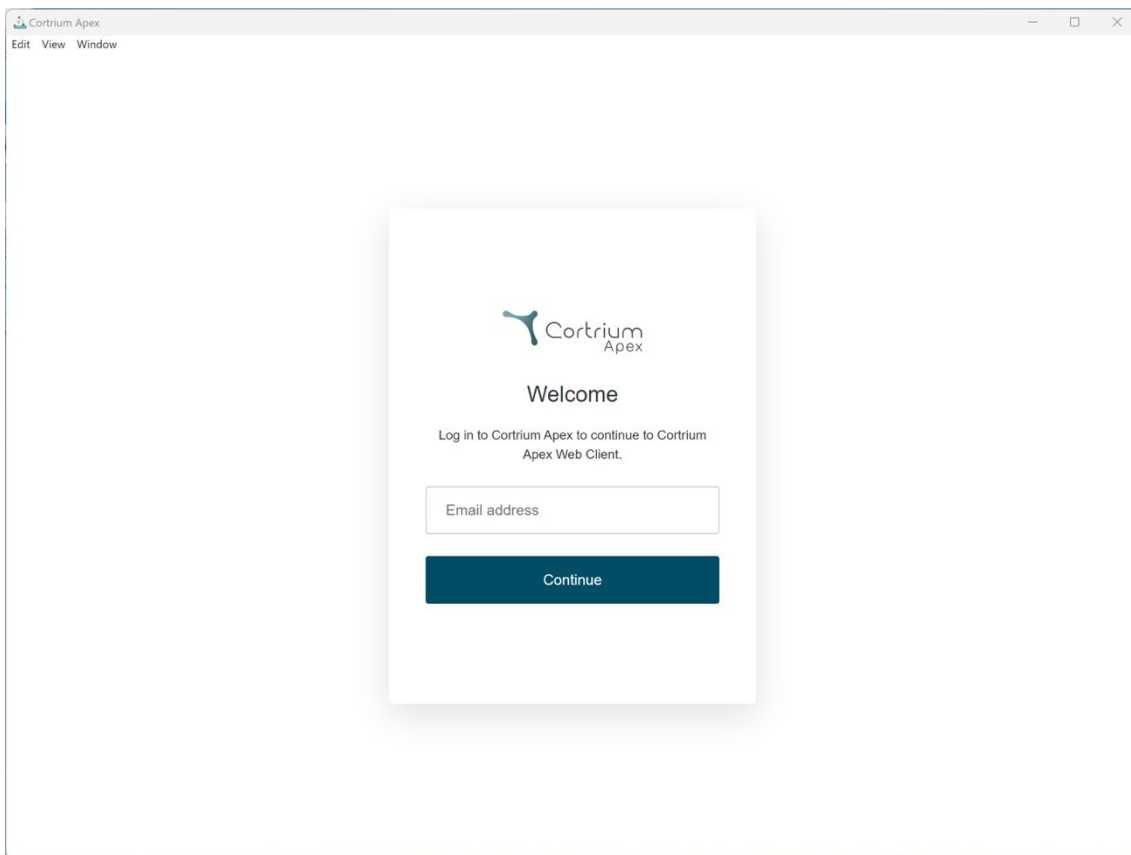
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". In the center of the window is a white login card with a shadow. The card features the Cortrium Apex logo at the top, followed by the heading "Welcome". Below the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." There is a text input field labeled "Email address" and a dark blue button labeled "Continue" below it.

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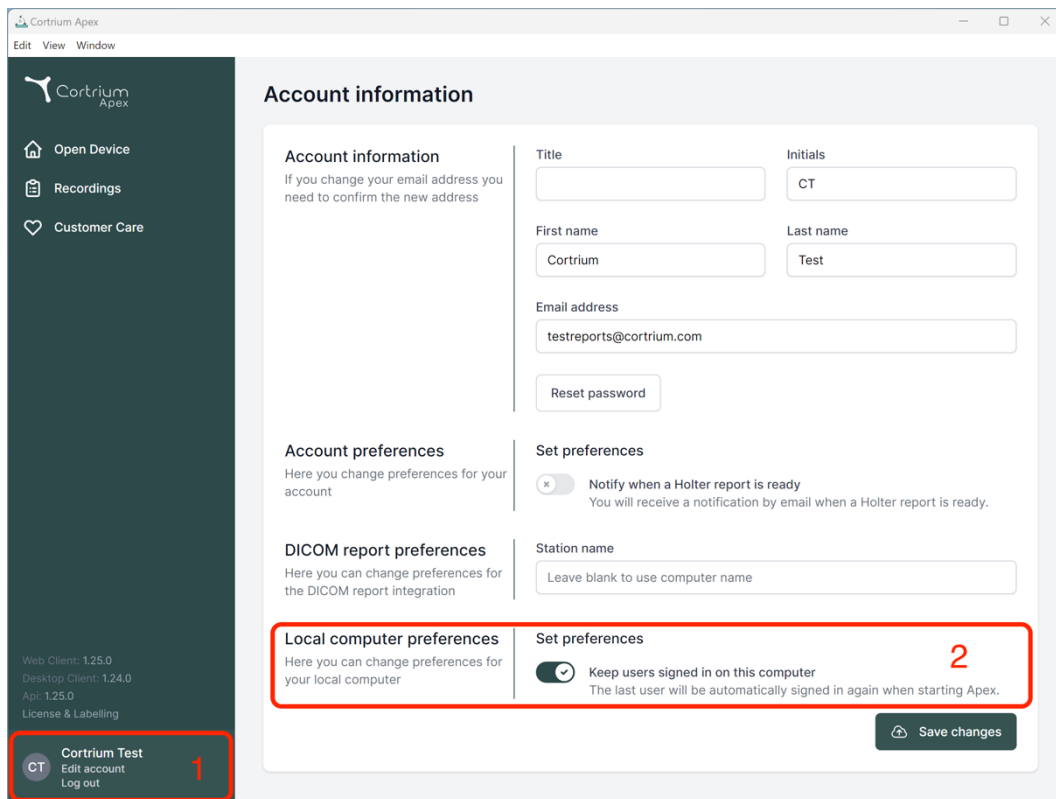The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these, it lists version information: 'Web Client: 1.25.0', 'Desktop Client: 1.24.0', 'Api: 1.25.0', and 'License & Labelling'. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with options to 'Edit account' and 'Log out', marked with a red '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' (with fields for Title, Initials, First name, Last name, and Email address), 'Account preferences' (with a toggle for 'Notify when a Holter report is ready'), 'DICOM report preferences' (with a field for 'Station name'), and 'Local computer preferences' (with a toggle for 'Keep users signed in on this computer'). The 'Local computer preferences' section is highlighted with a red box and a red '2'. A 'Save changes' button is located at the bottom right of the main content area.

Account information

If you change your email address you need to confirm the new address

Title: Initials:

First name: Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

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DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

Local computer preferences

Here you can change preferences for your local computer

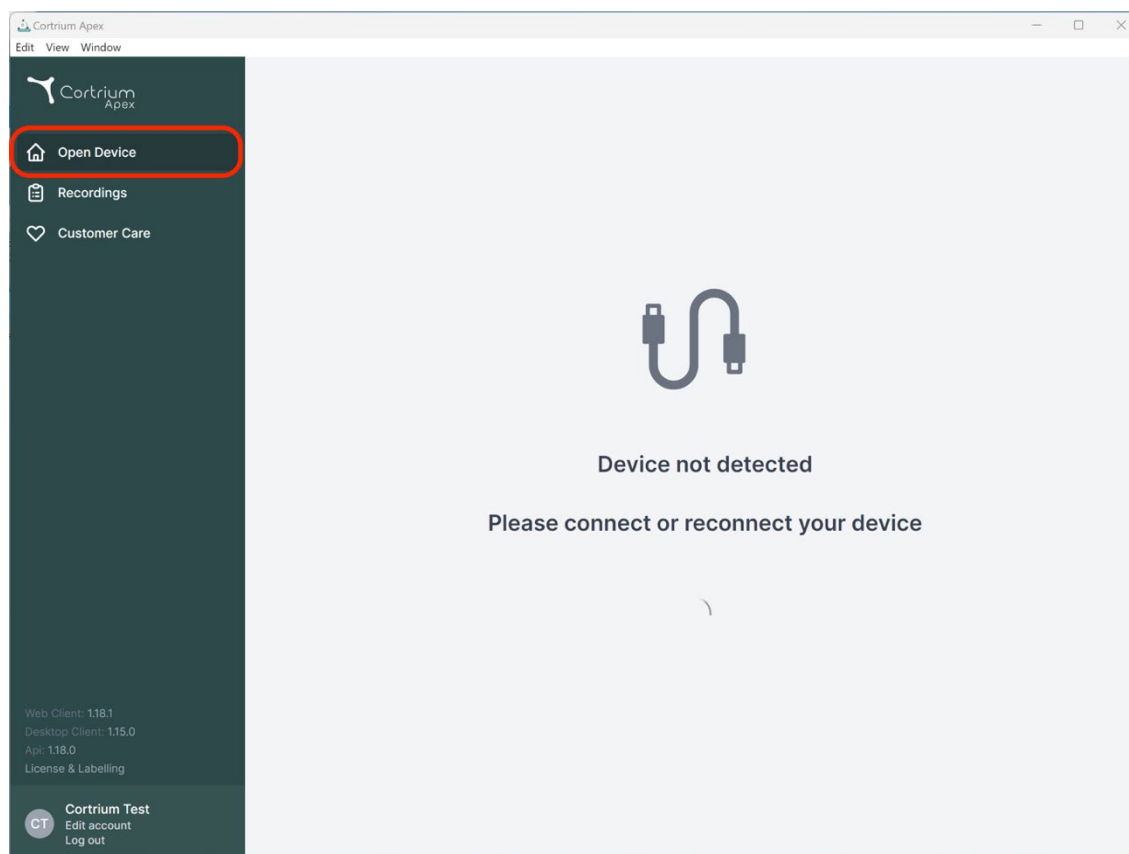
☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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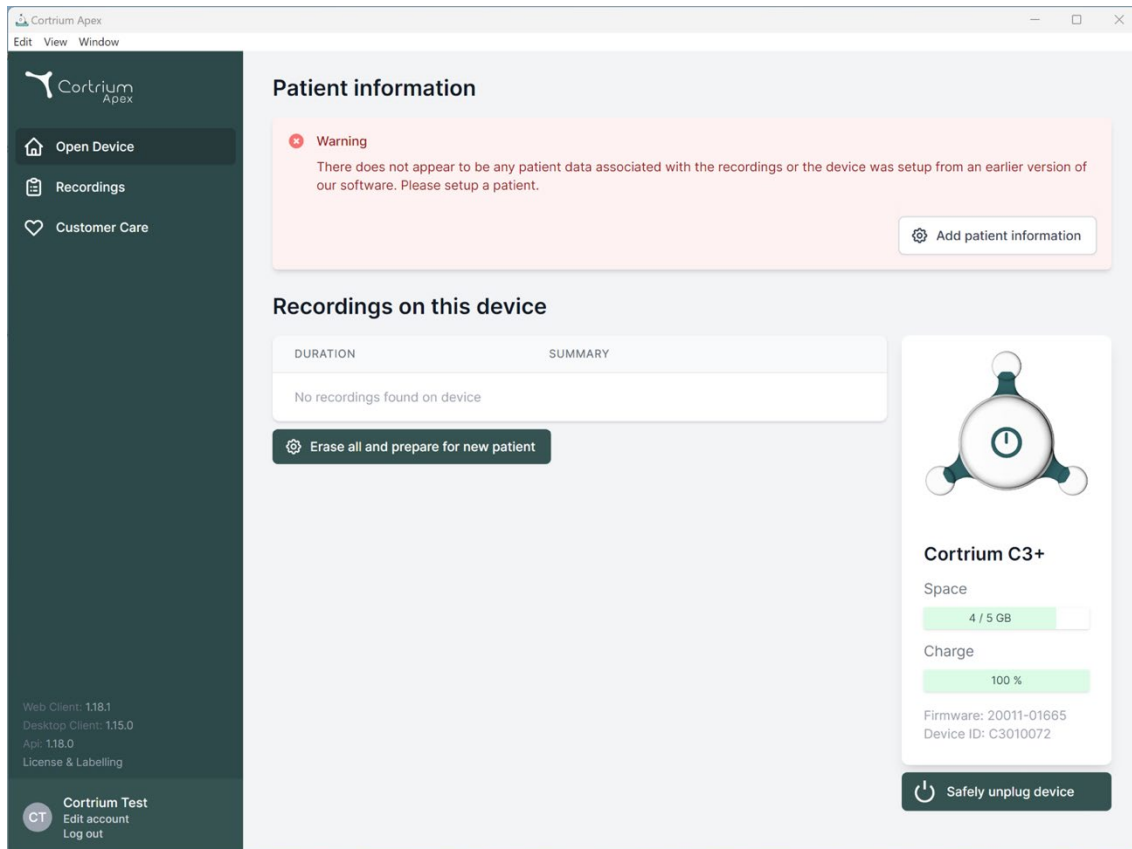
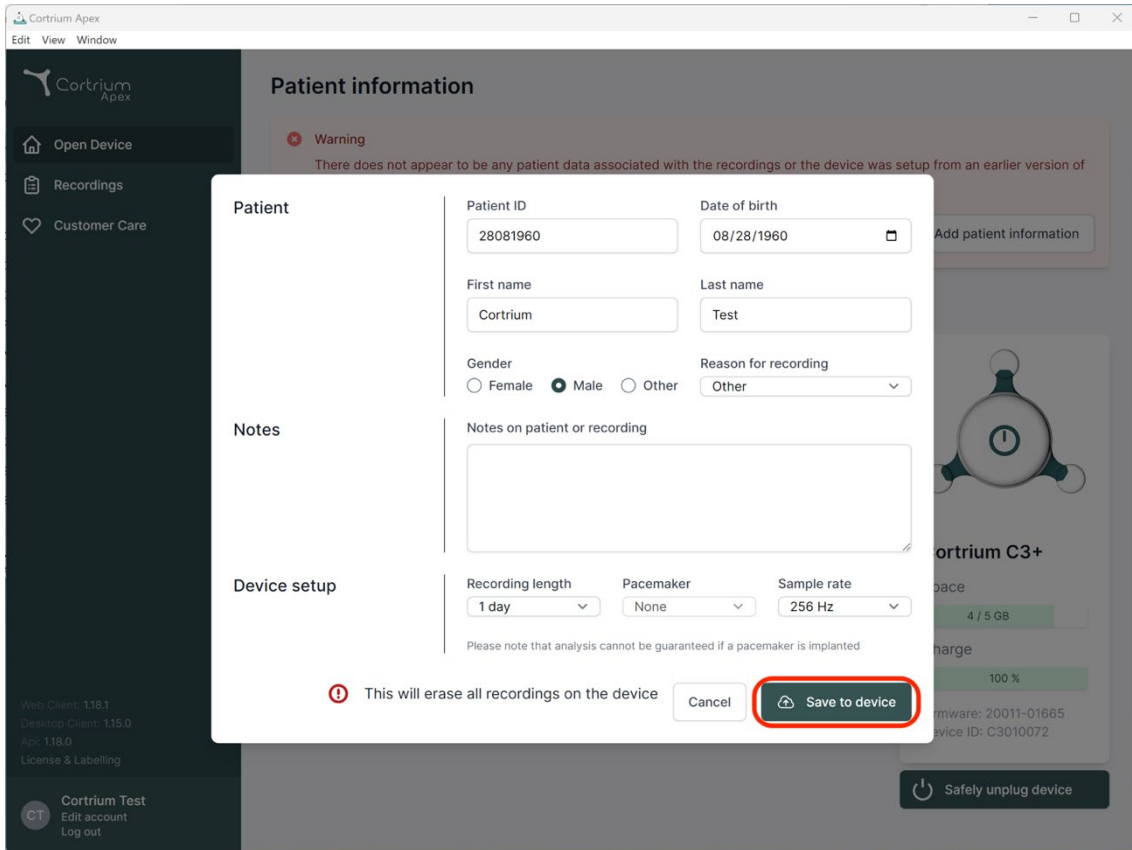
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
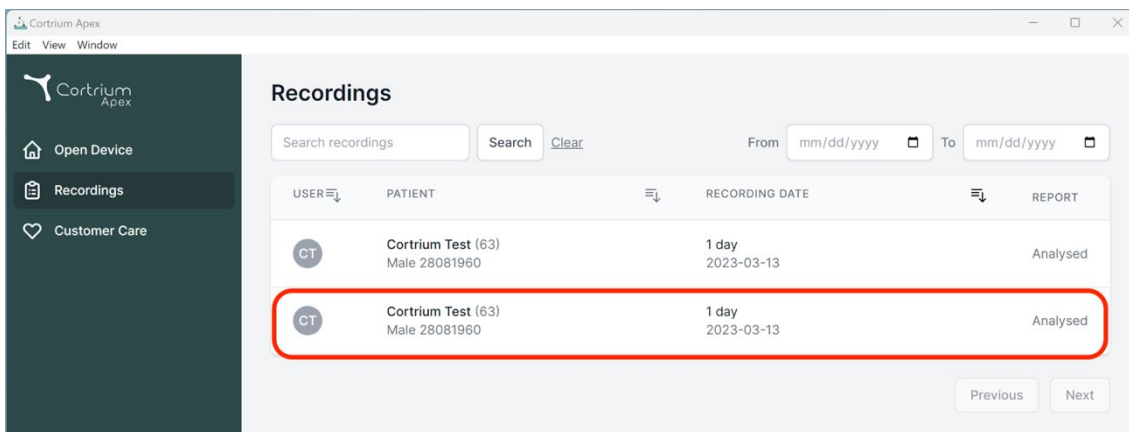
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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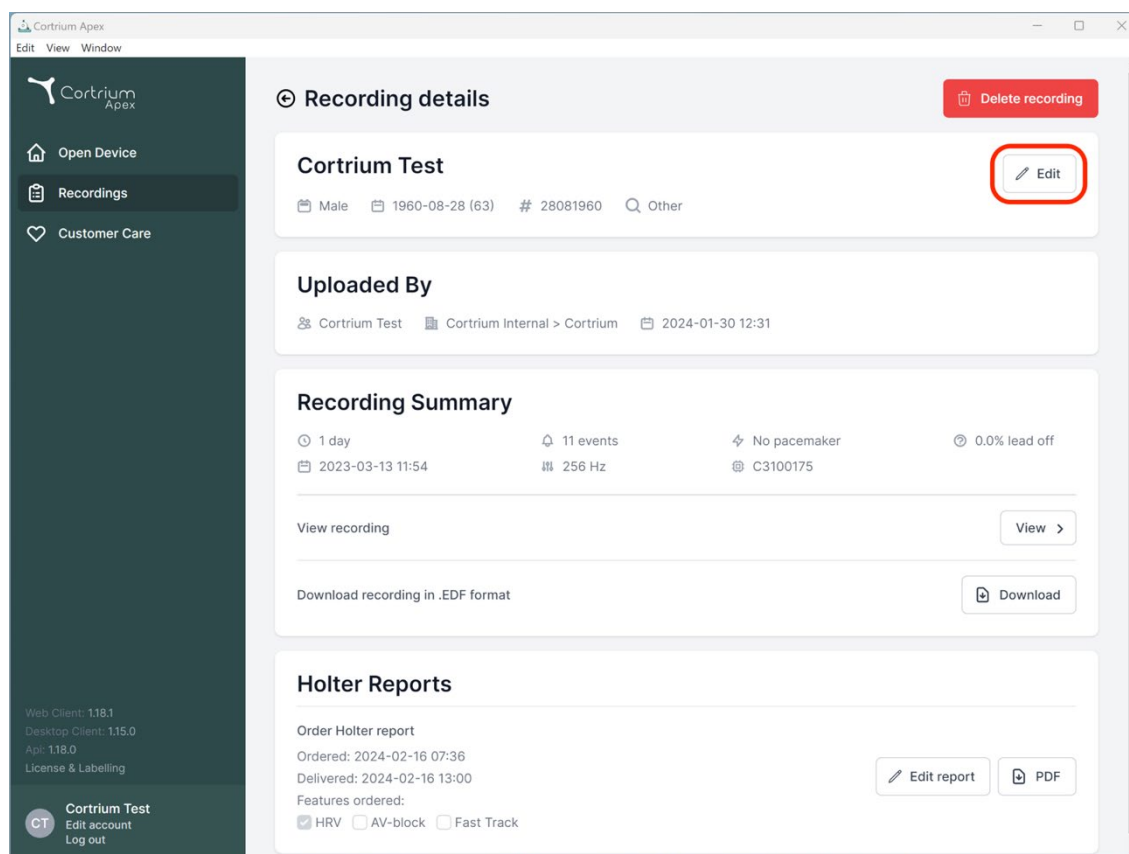2023-03-13 11:54 256 Hz C3100175

View recording View

Download recording in .EDF format Download

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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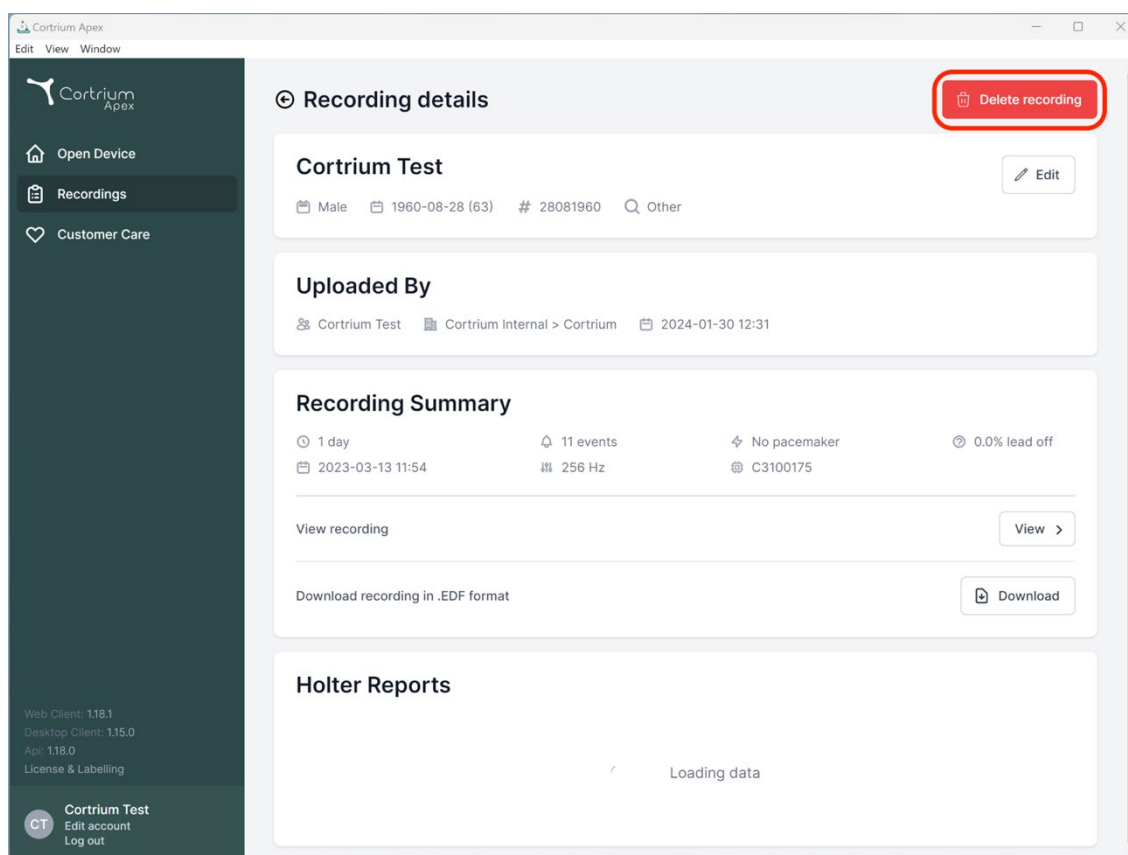
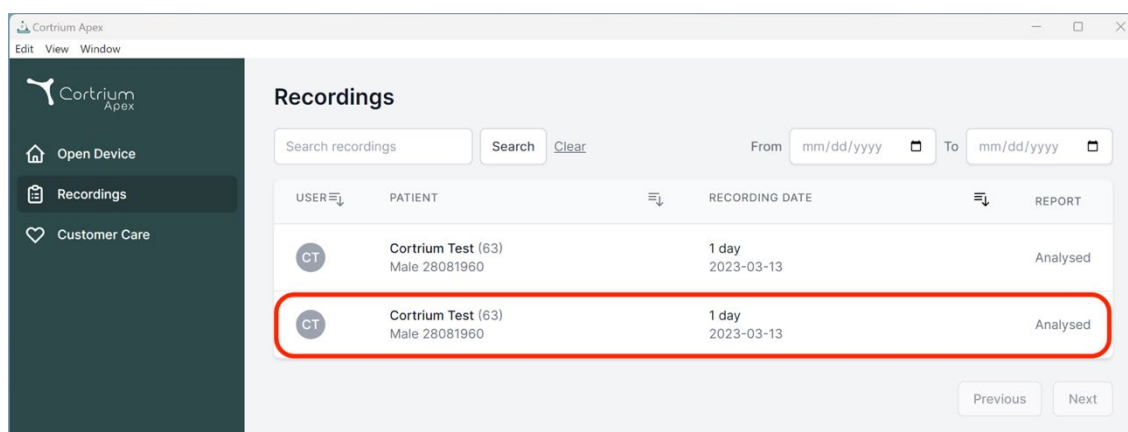
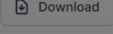
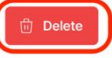
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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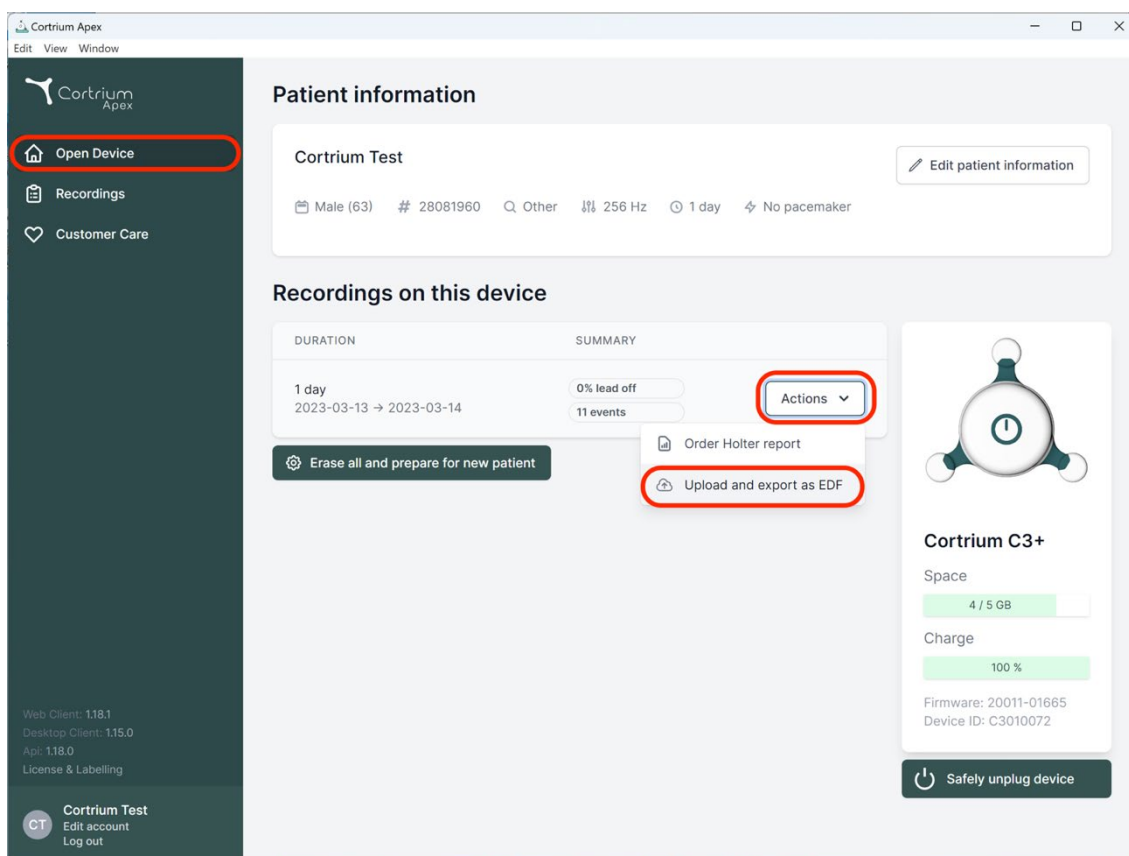
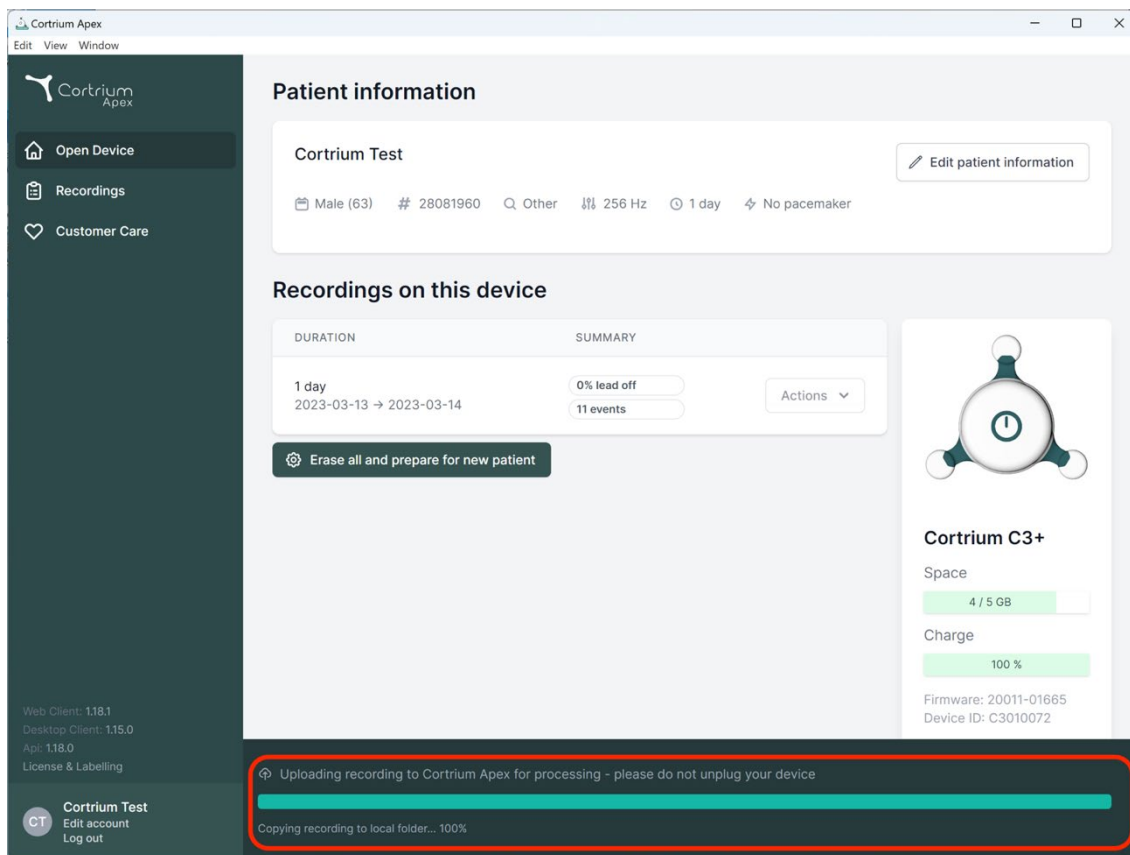
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

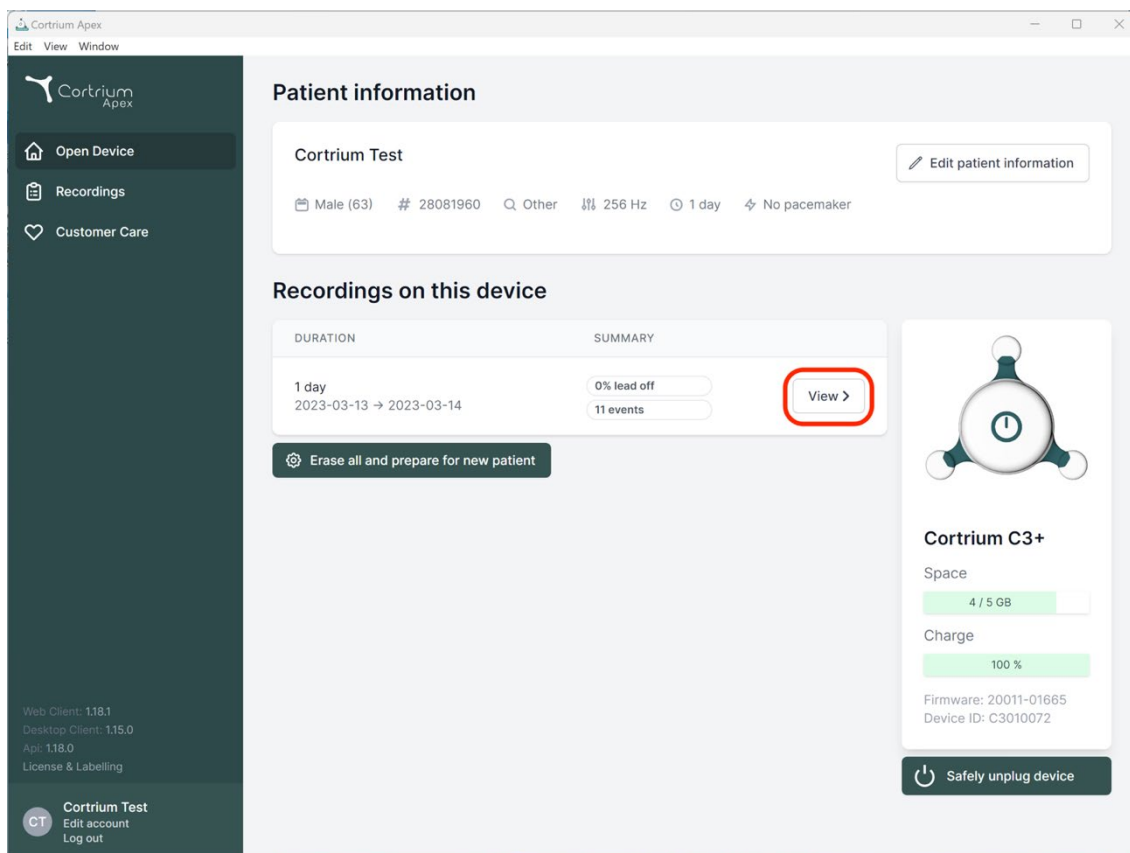
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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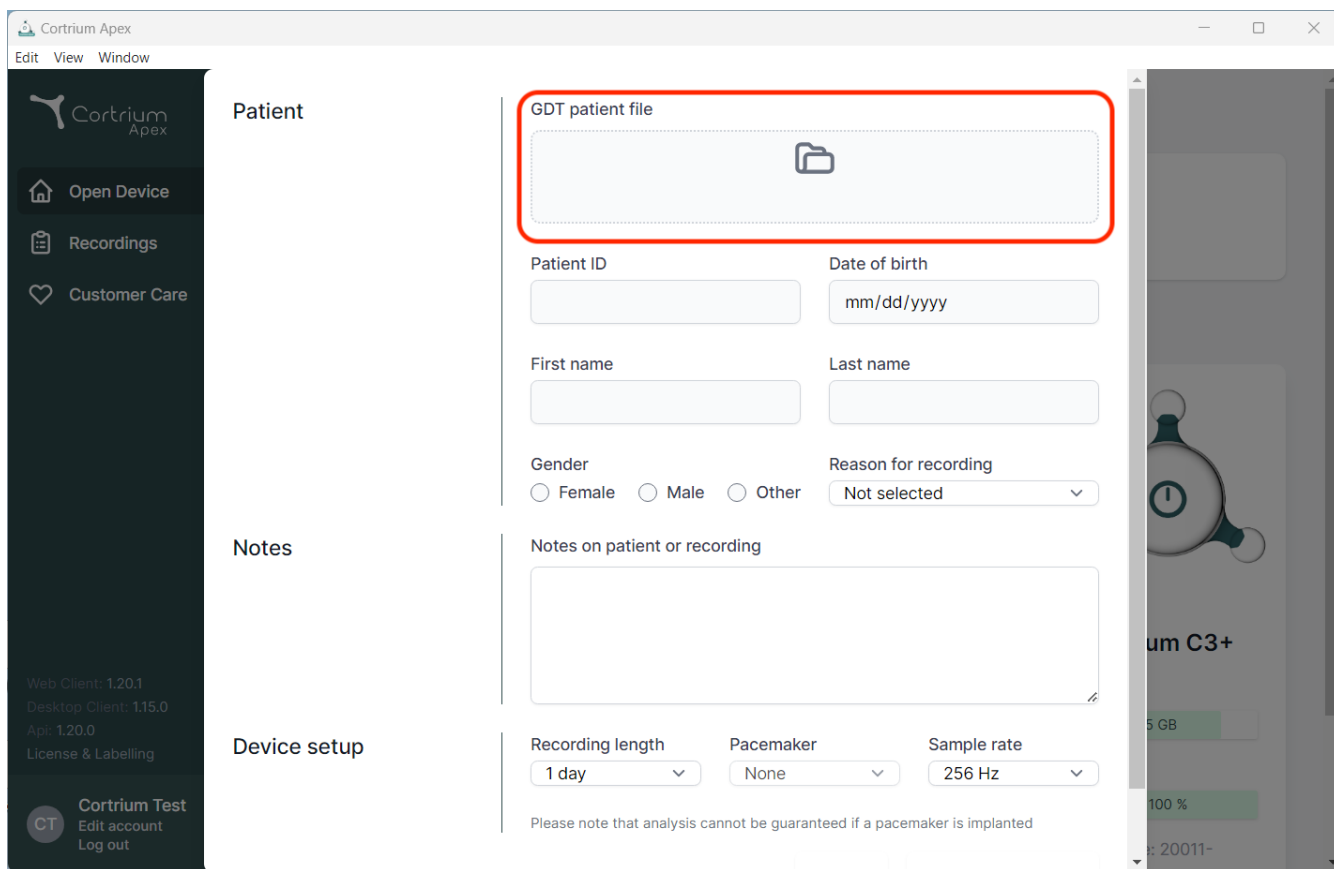
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

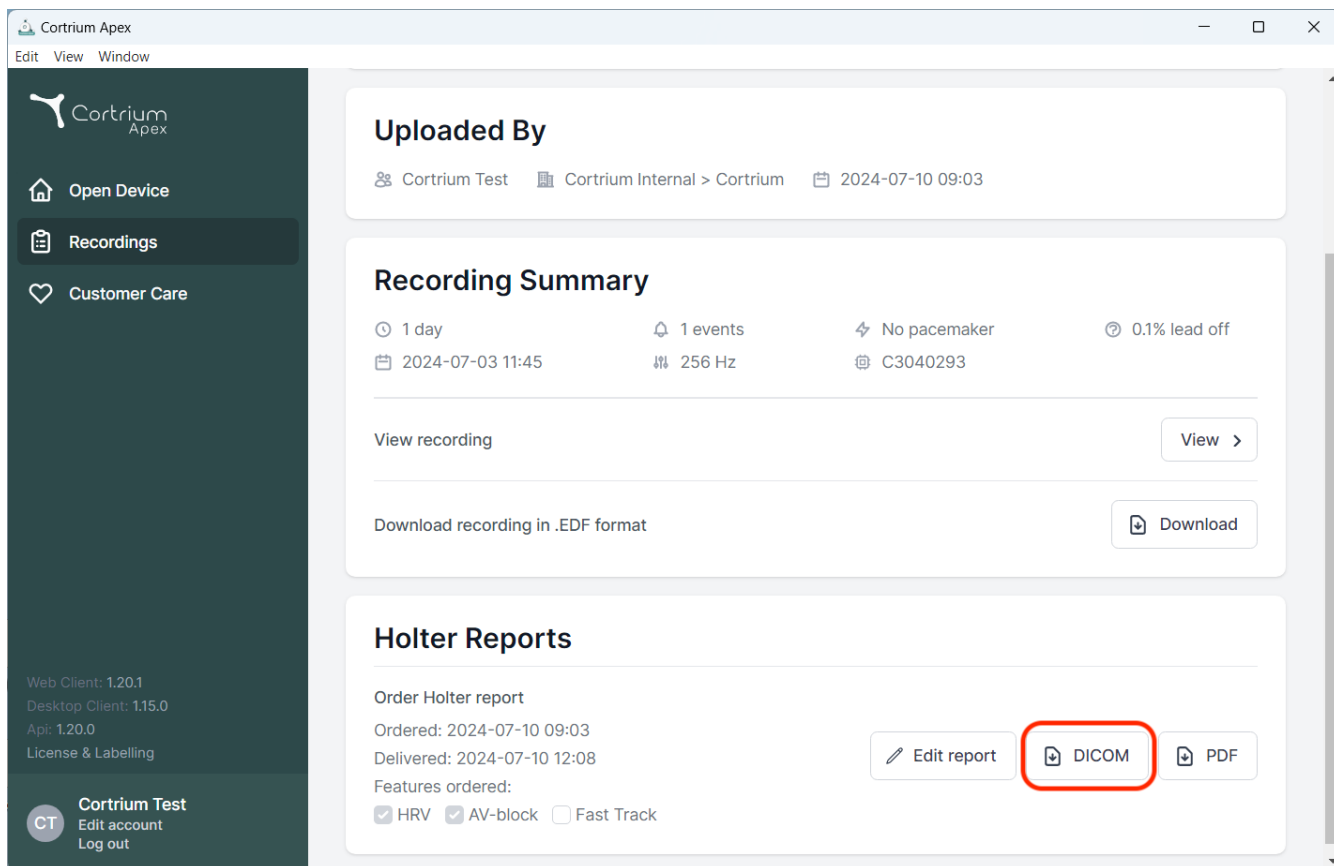
The main content area is titled "Patient" and contains a form with the following sections:

- GDT patient file:** A red-bordered box containing a file upload icon.
- Patient ID:** A text input field.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes:** A section titled "Notes on patient or recording" with a large text area.
- Device setup:** A section with three dropdown menus: "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the "Device setup" section, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By' (showing 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'), 'Recording Summary' (displaying device details like '1 day', '1 events', 'No pacemaker', '0.1% lead off', and '256 Hz'), and 'Holter Reports'. In the 'Holter Reports' section, under 'Order Holter report', there are buttons for 'Edit report', 'DICOM' (which is circled in red), and 'PDF'. Below these buttons, the report details show 'Ordered: 2024-07-10 09:03', 'Delivered: 2024-07-10 12:08', and 'Features ordered: HRV, AV-block, Fast Track'.

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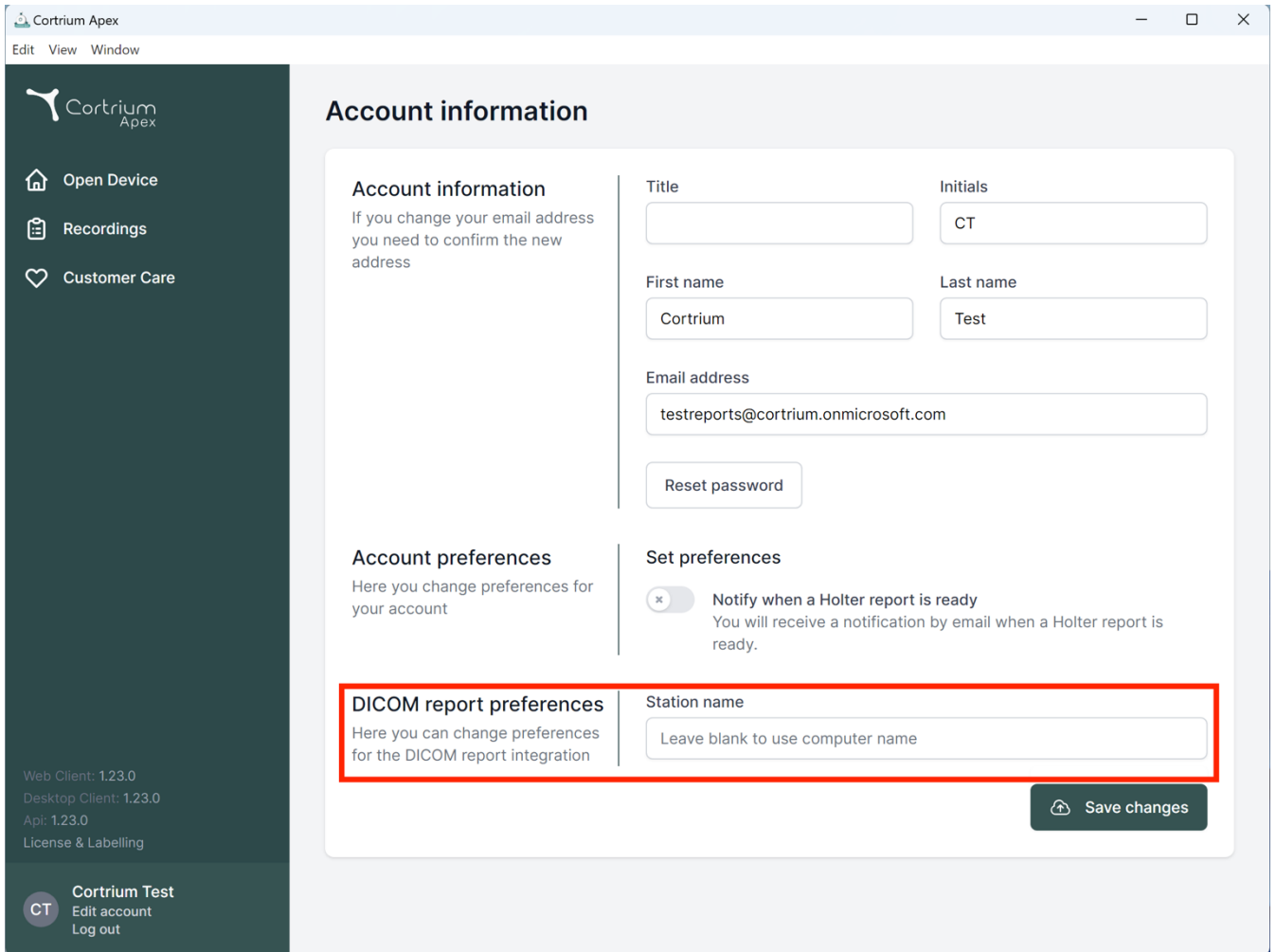Edit View Window

Account information

Account information
If you change your email address you need to confirm the new address

Title

Initials

First name

Last name

Email address

[Reset password](#)

Account preferences
Here you change preferences for your account

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name

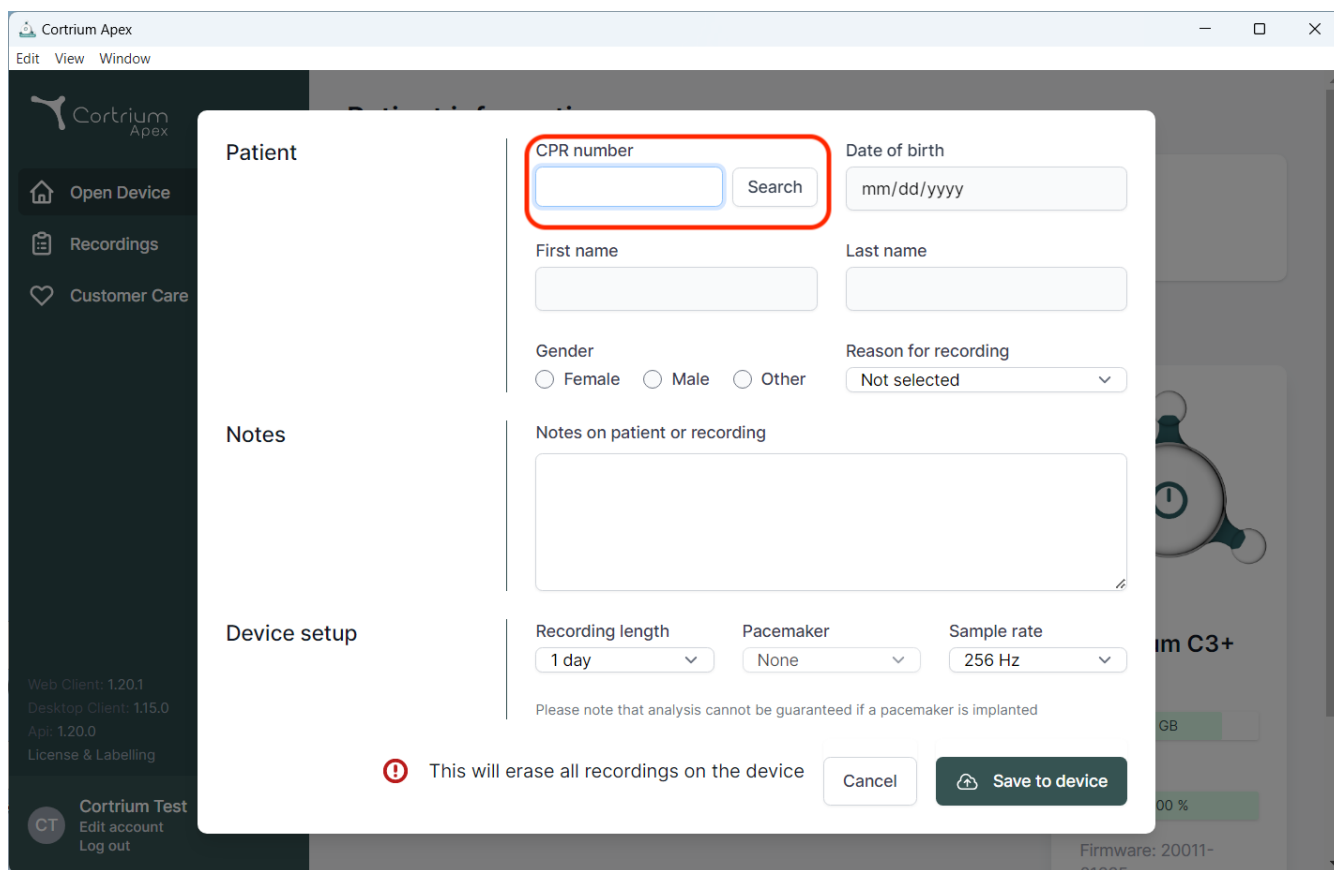
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, allowing for patient search and data entry. The form includes the following fields and options:

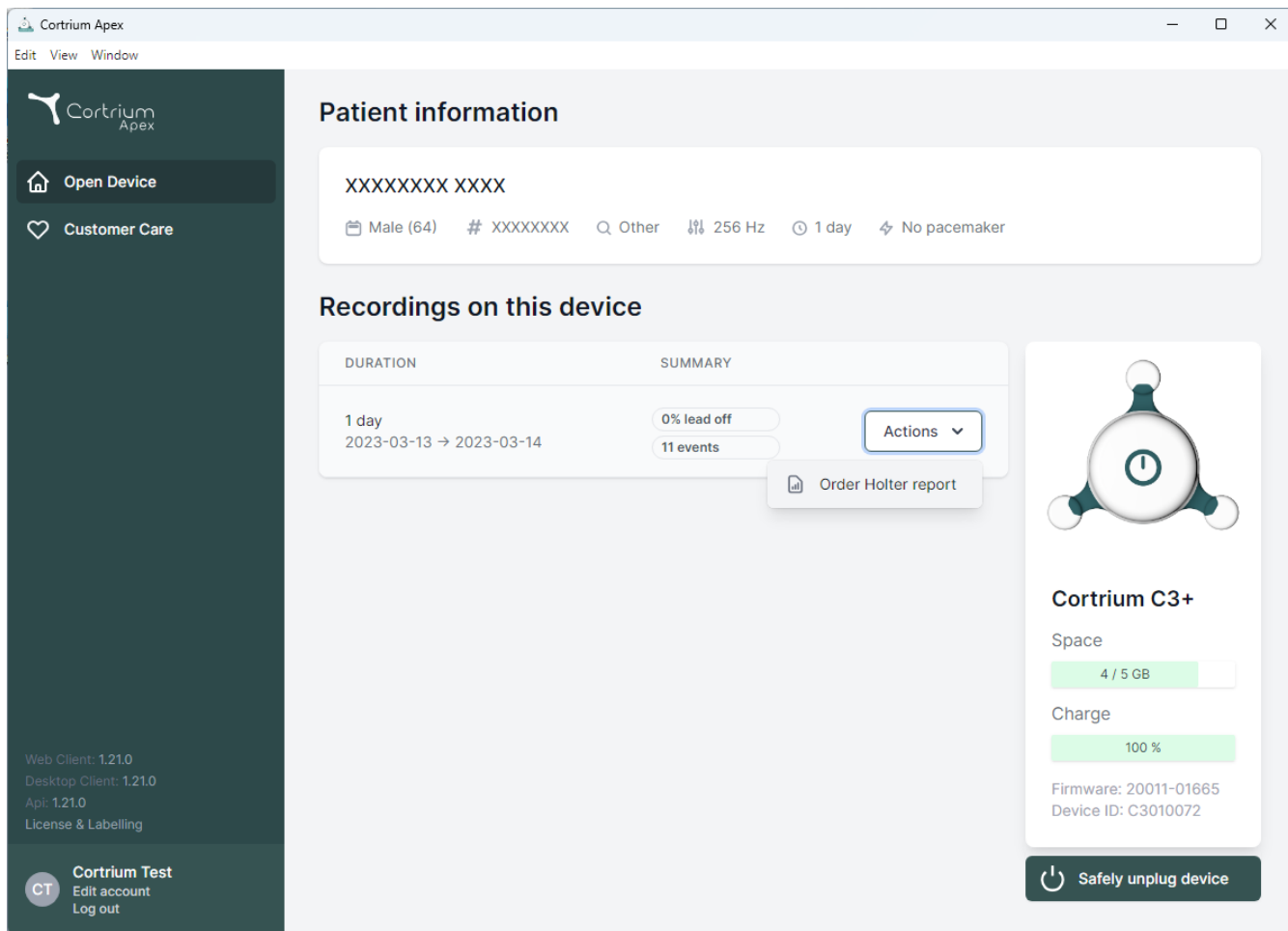
- CPR number:** A text input field with a blue border, highlighted by a red rectangle, and a "Search" button next to it.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** A section containing three dropdown menus: "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup section, a warning message is displayed: "⚠ This will erase all recordings on the device". At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

The background shows the Cortrium Apex application menu with options like "Open Device", "Recordings", and "Customer Care". The bottom left corner includes version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", and "App: 1.20.0".

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device' (with a house icon), and 'Customer Care' (with a heart icon). Below these are version details: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', and 'License & Labelling'. At the bottom of the sidebar is a 'Cortrium Test' section with a 'CT' icon, 'Edit account', and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section contains a table with two columns: 'DURATION' and 'SUMMARY'. The table has one entry for '1 day' (2023-03-13 → 2023-03-14) with a summary of '0% lead off' and '11 events'. An 'Actions' dropdown menu is open, showing the option 'Order Holter report'. On the right, a device status card for 'Cortrium C3+' shows 'Space' at '4 / 5 GB' and 'Charge' at '100 %'. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

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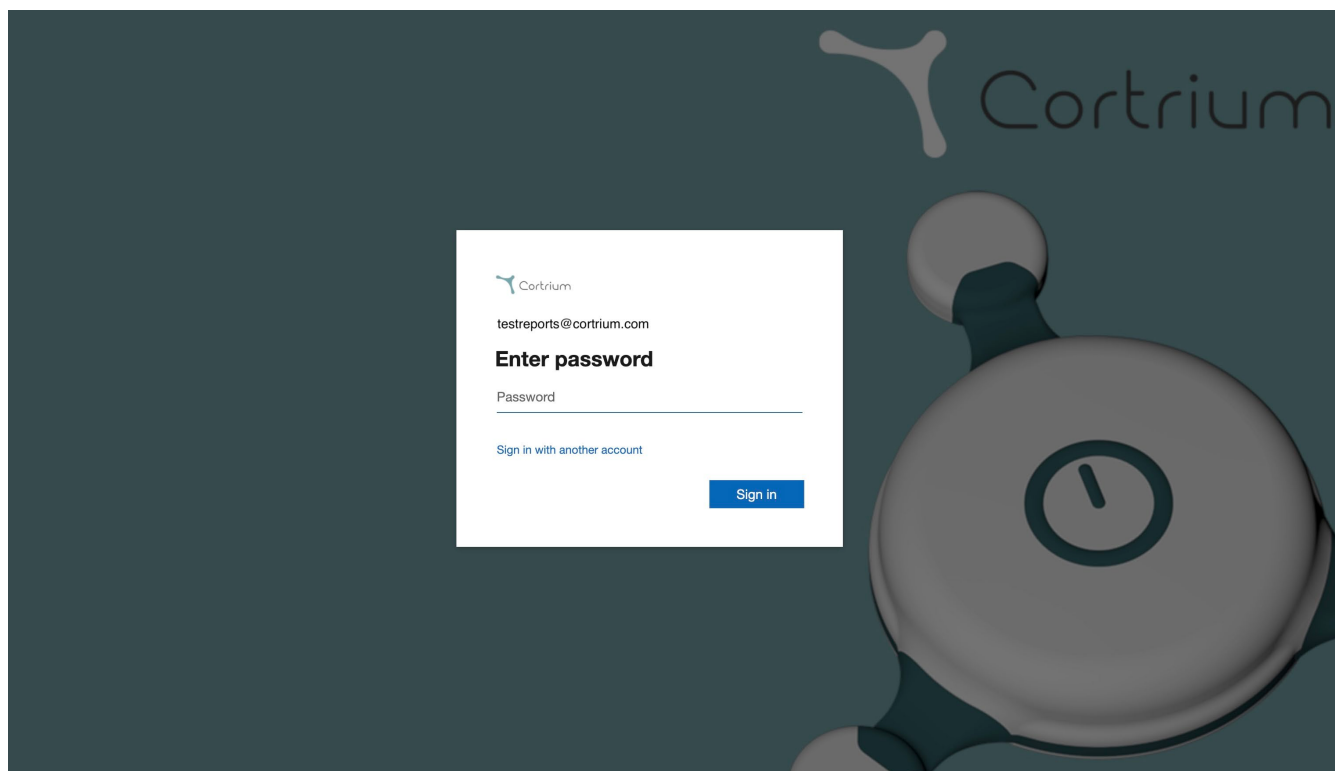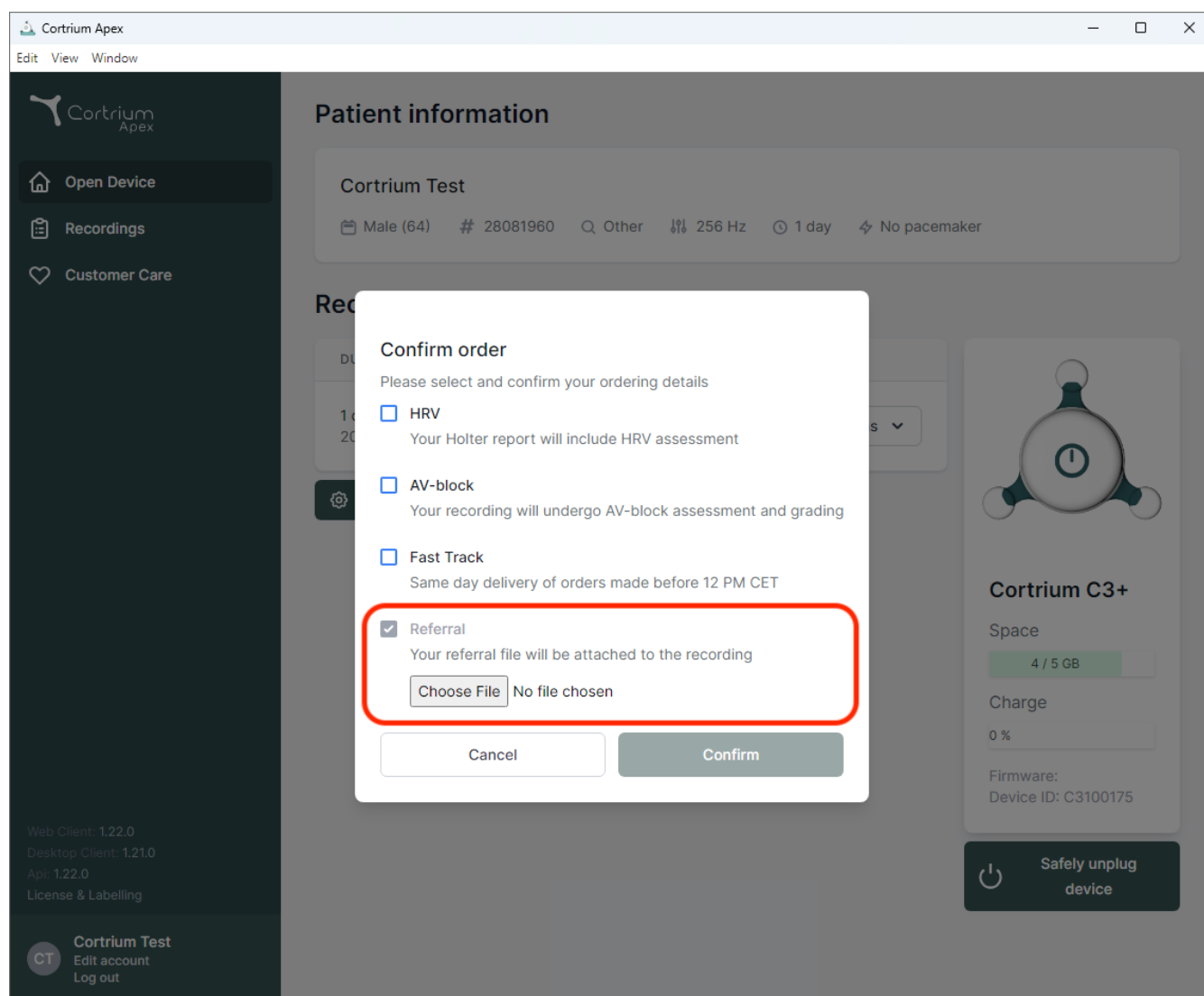
Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off

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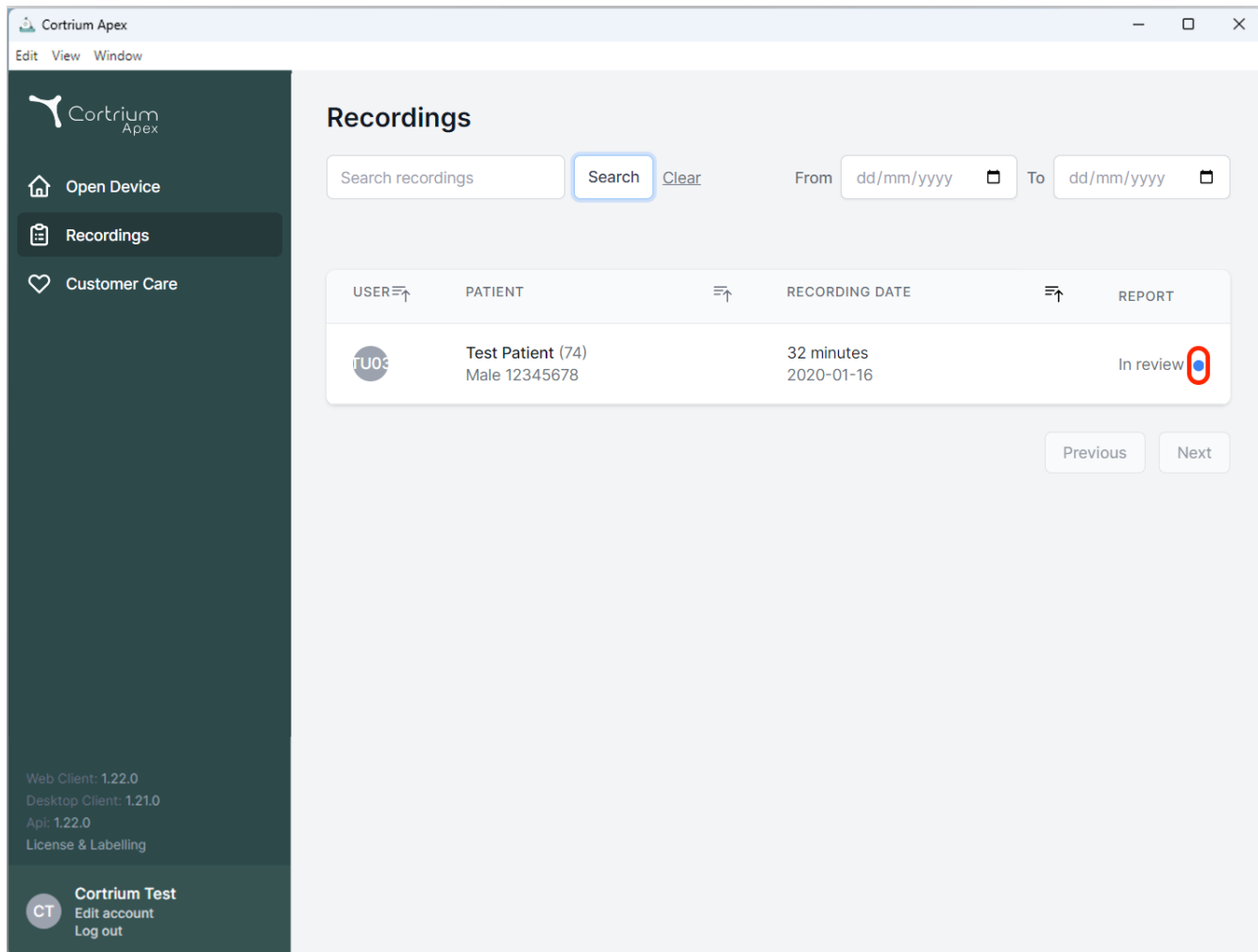
Recordings

Search recordings [Clear](#) From To

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review ●

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
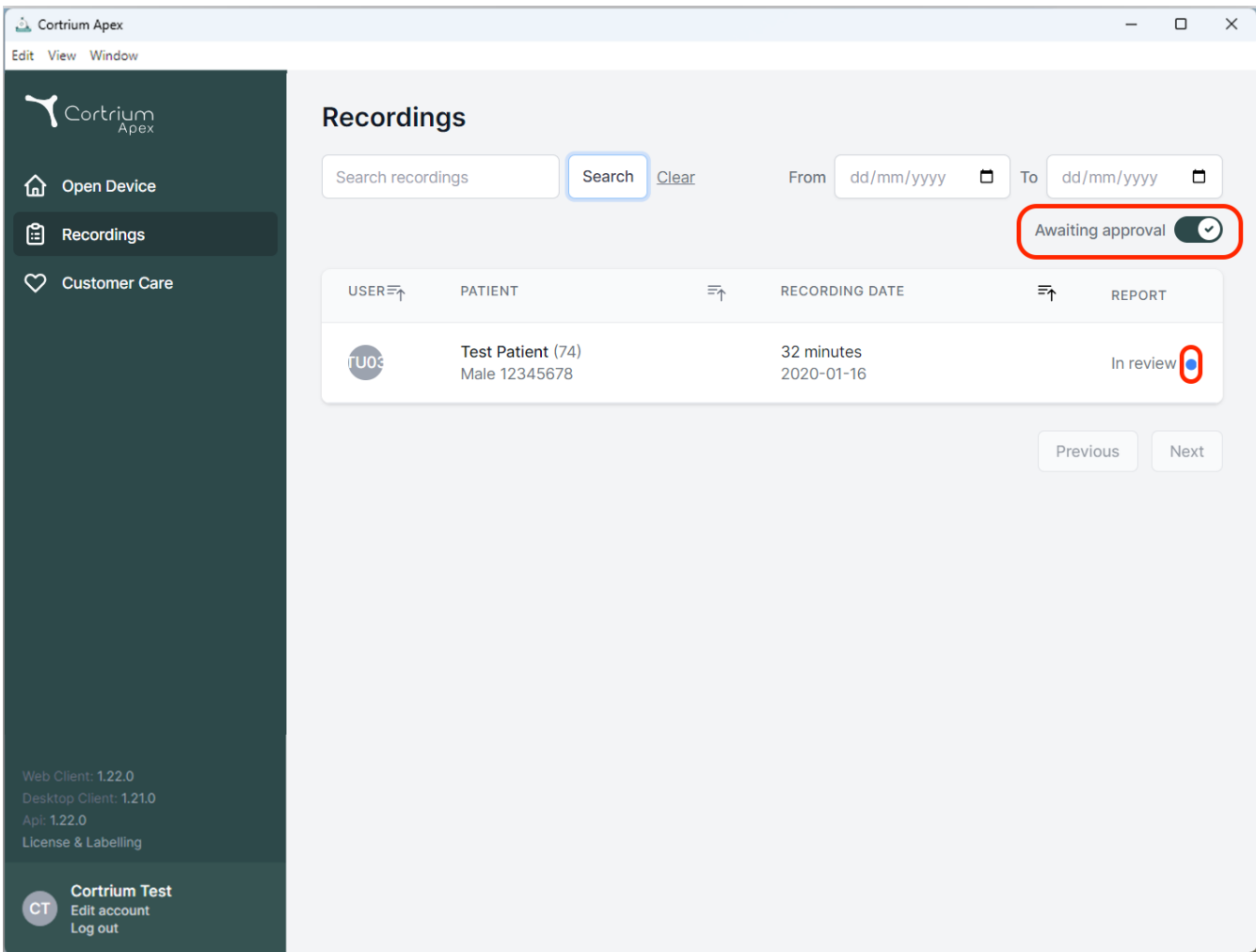
From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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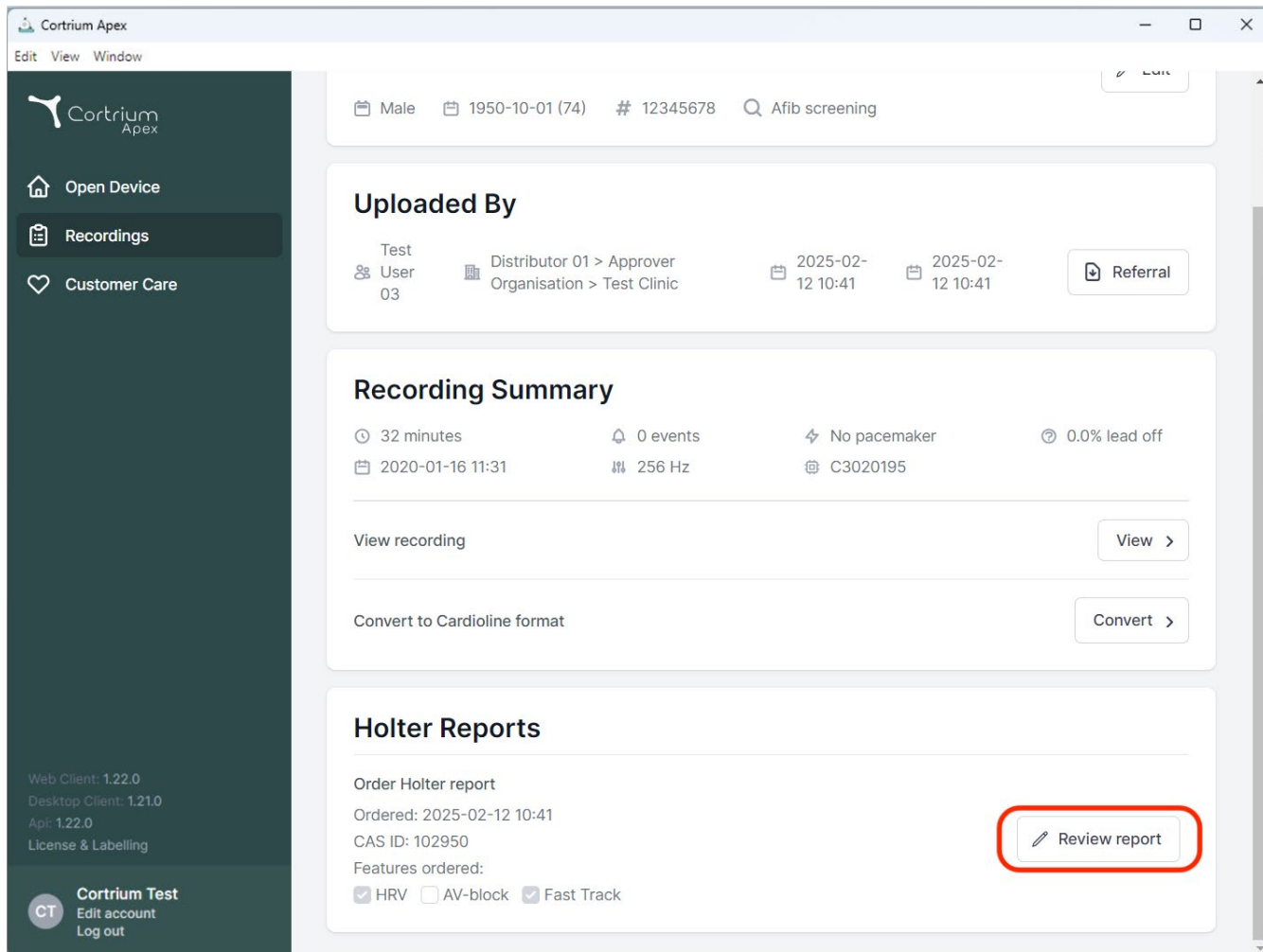The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: "Open Device", "Recordings" (highlighted), and "Customer Care". Below these are version details (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0) and a "Cortrium Test" section with "Edit account" and "Log out" options. The main content area shows patient information: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this is the "Uploaded By" section, followed by a "Recording Summary" with details like 32 minutes, 0 events, and 256 Hz. At the bottom is the "Holter Reports" section, which includes an "Order Holter report" button and a "Review report" button highlighted with a red rectangle. Other buttons like "Referral", "View", and "Convert" are also visible.

Cortrium Apex

Edit View Window

Male 1950-10-01 (74) # 12345678 Afib screening

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

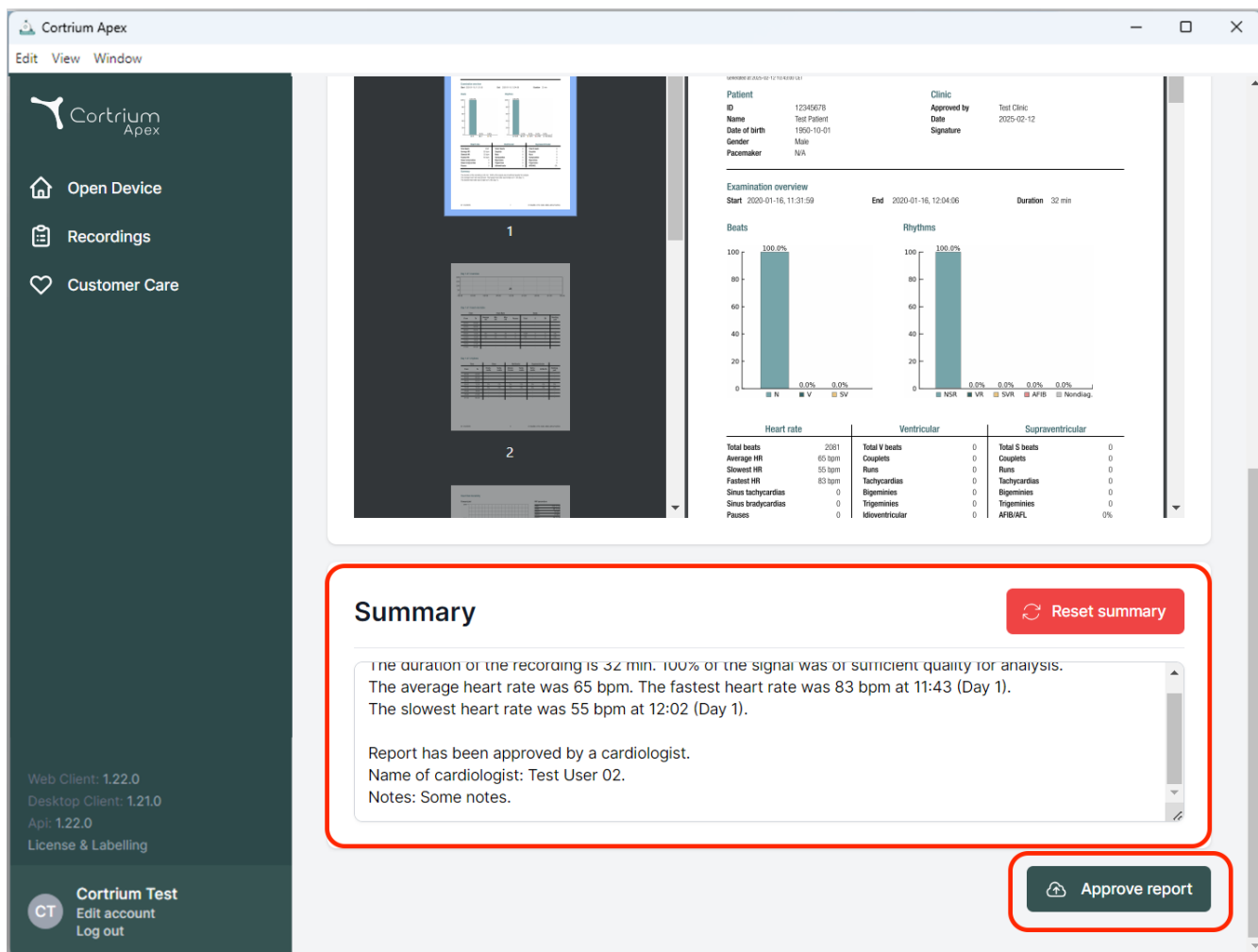
Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

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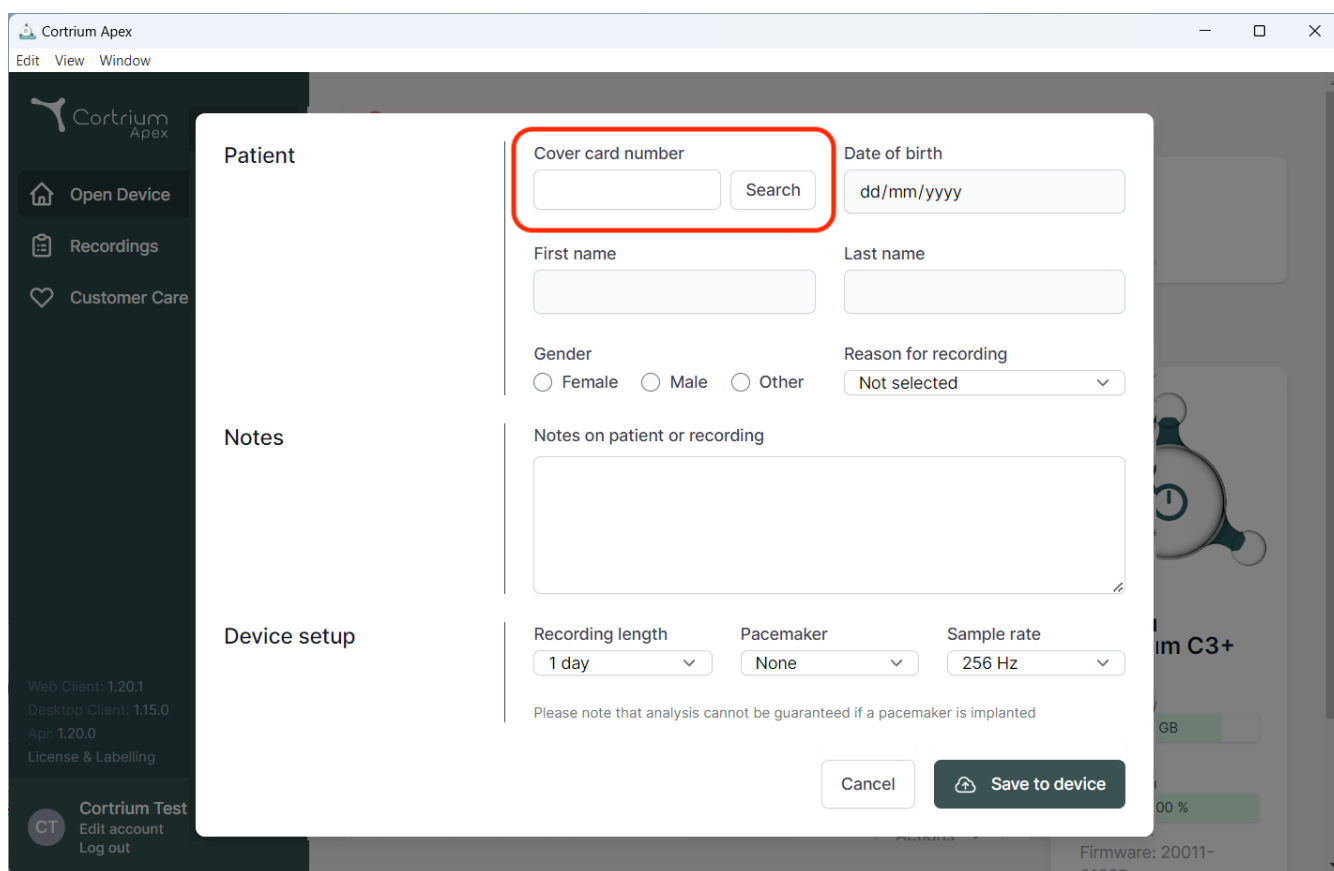
The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, allowing for patient search and recording configuration. The modal is divided into three main sections: "Patient", "Notes", and "Device setup".

Patient Section:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangle.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Three radio buttons labeled "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".

Notes Section:

- Notes on patient or recording:** A large text area for entering notes.

Device setup Section:

- Recording length:** A dropdown menu showing "1 day".
- Pacemaker:** A dropdown menu showing "None".
- Sample rate:** A dropdown menu showing "256 Hz".

Below the device setup options, a note states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

The background interface shows a sidebar with navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar contains version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0, License & Labelling) and user options (Cortrium Test, Edit account, Log out).

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dania
E-mail: info@cortrium.com • Strona internetowa: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
47 / 47