Cortrium Apex 1.27

Cortrium Apex 1.27

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
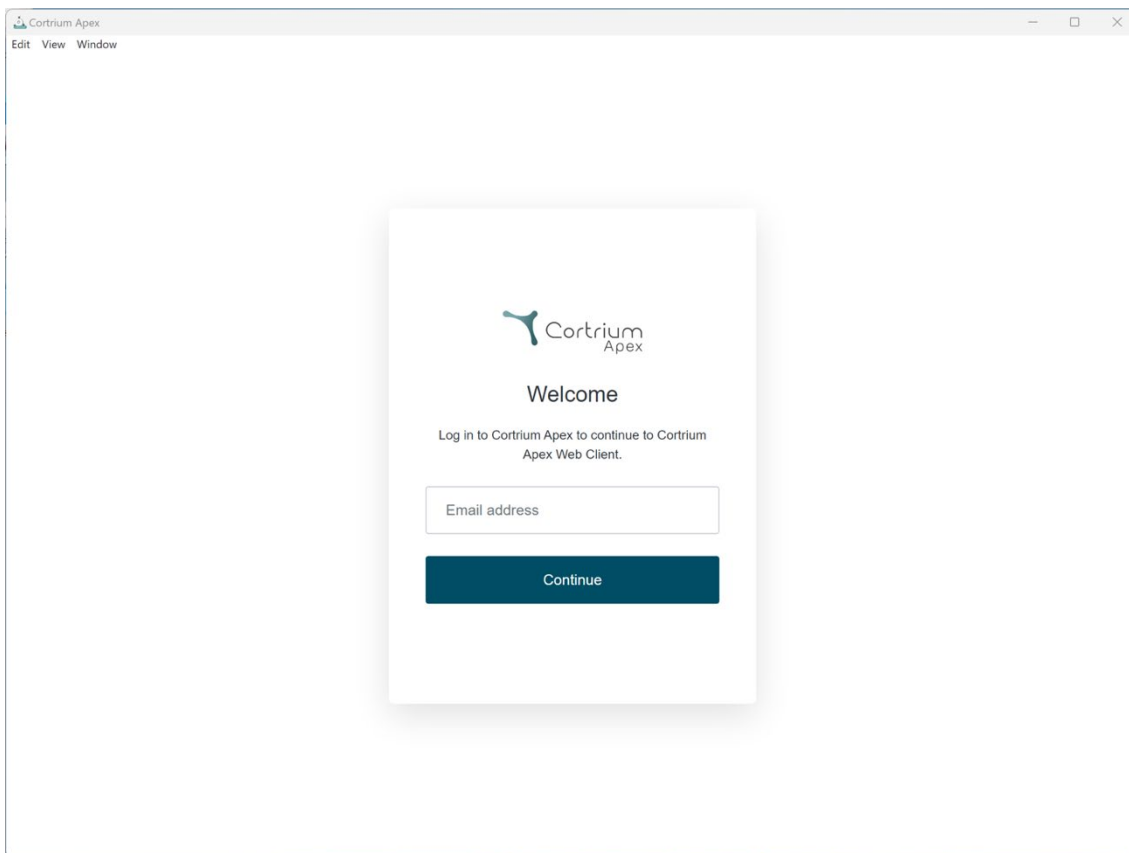
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". The main content area displays a login form centered on the page. The form has a white background and a subtle drop shadow. At the top of the form is the Cortrium Apex logo. Below the logo, the word "Welcome" is displayed in a bold font. Underneath, a smaller line of text reads "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." The form contains a single text input field labeled "Email address" and a dark blue button labeled "Continue" positioned below the input field.

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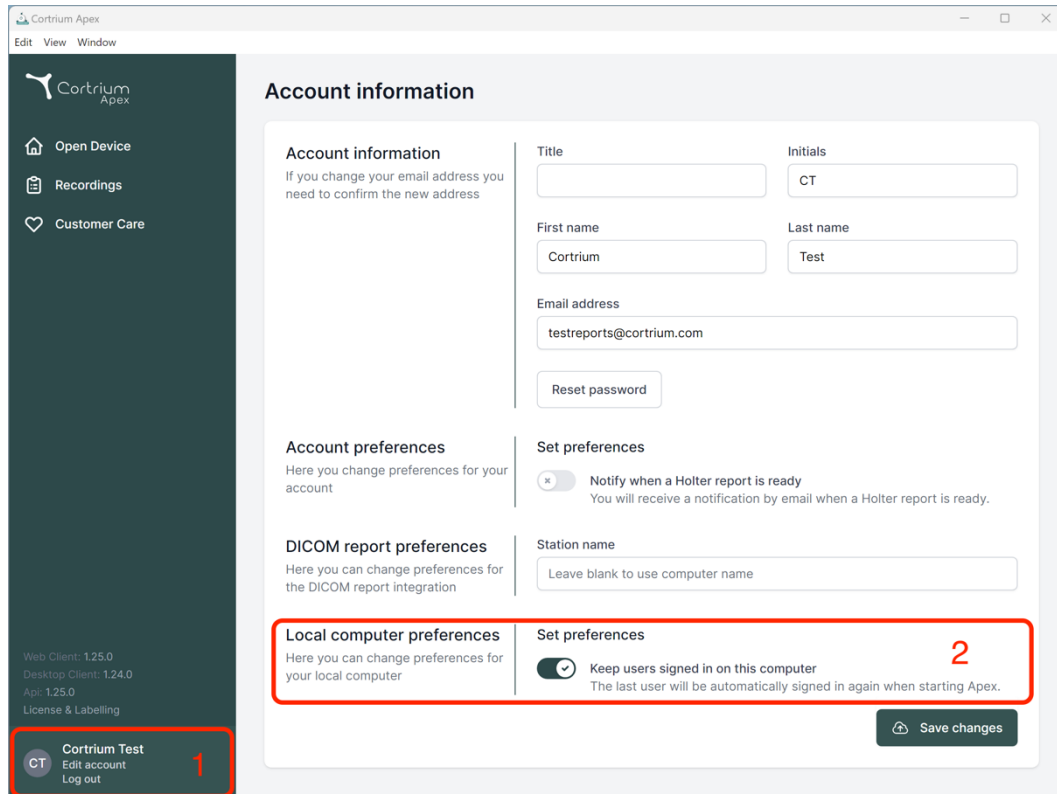Account information

Account information
If you change your email address you need to confirm the new address

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences
Here you change preferences for your account

DICOM report preferences
Here you can change preferences for the DICOM report integration

Local computer preferences
Here you can change preferences for your local computer

Set preferences

☒ **Notify when a Holter report is ready**
You will receive a notification by email when a Holter report is ready.

Station name

Set preferences

☒ **Keep users signed in on this computer**
The last user will be automatically signed in again when starting Apex.

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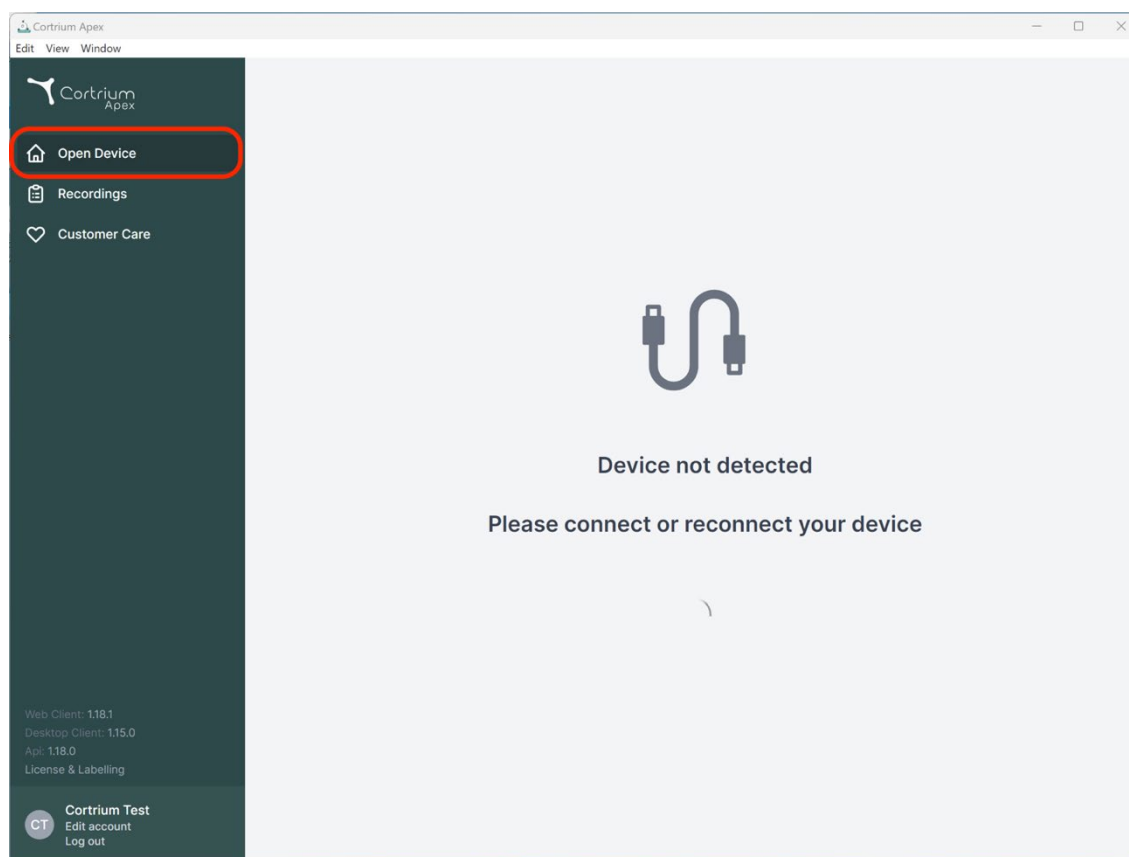
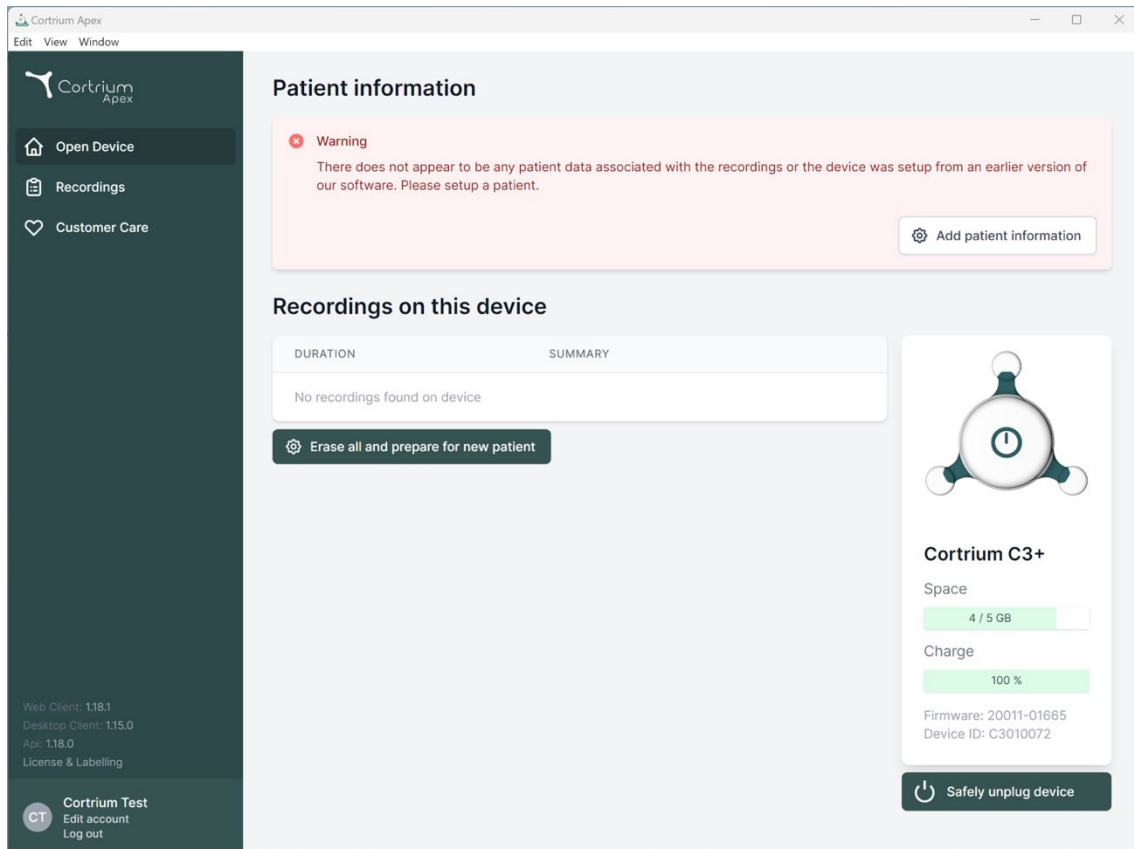
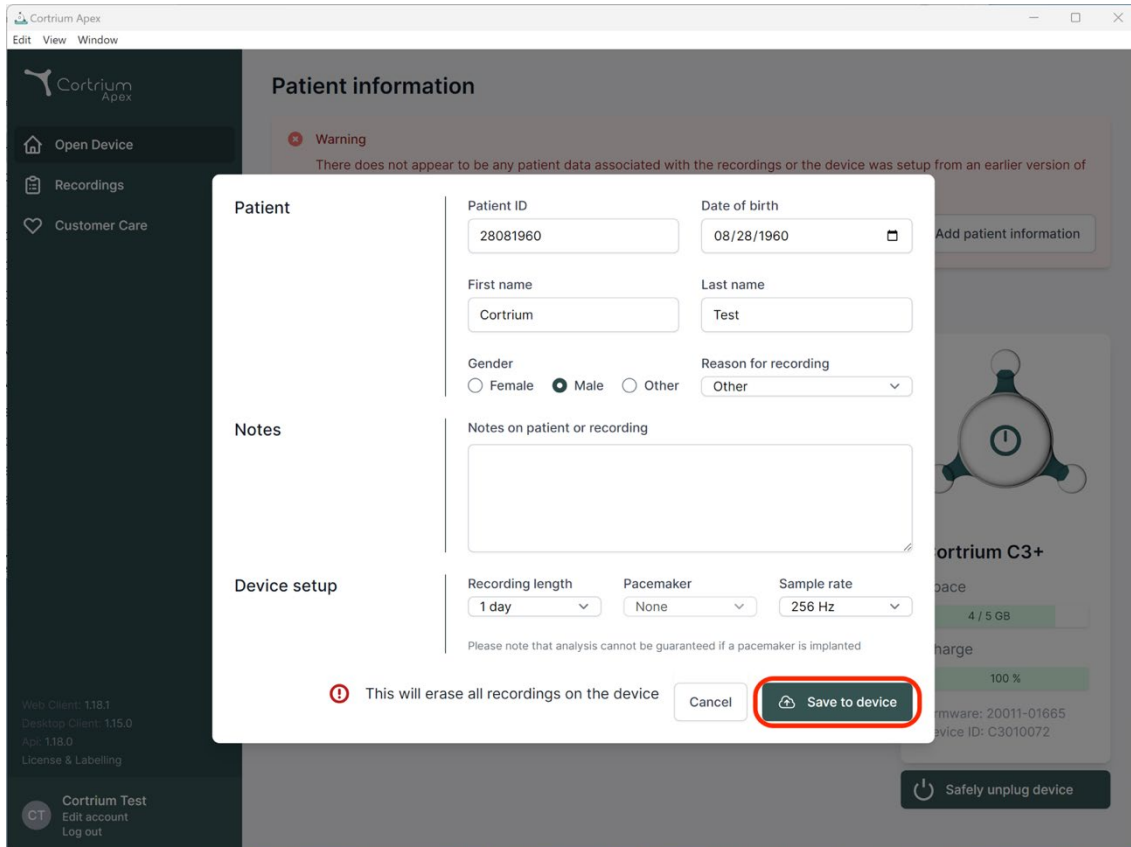
Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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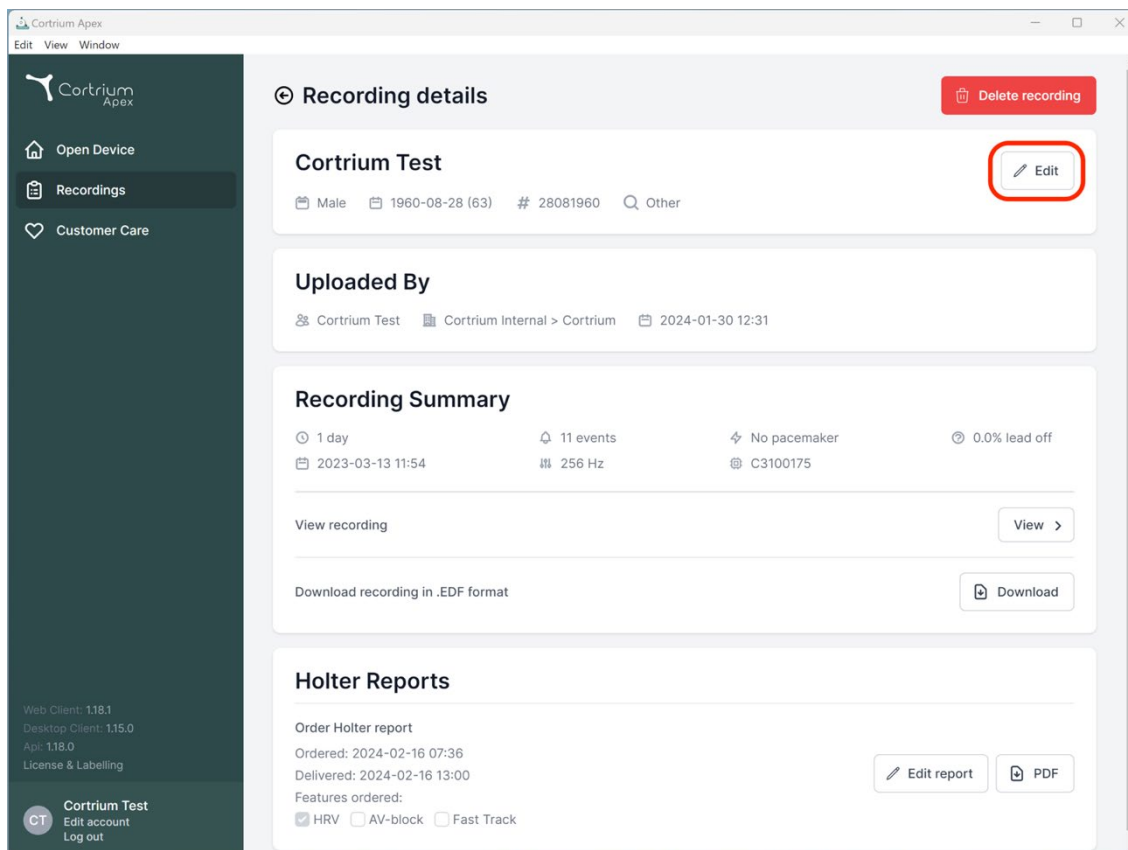
Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary
 1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports
 Order Holter report
 Ordered: 2024-02-16 07:36
 Delivered: 2024-02-16 13:00
 Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track Edit report PDF

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
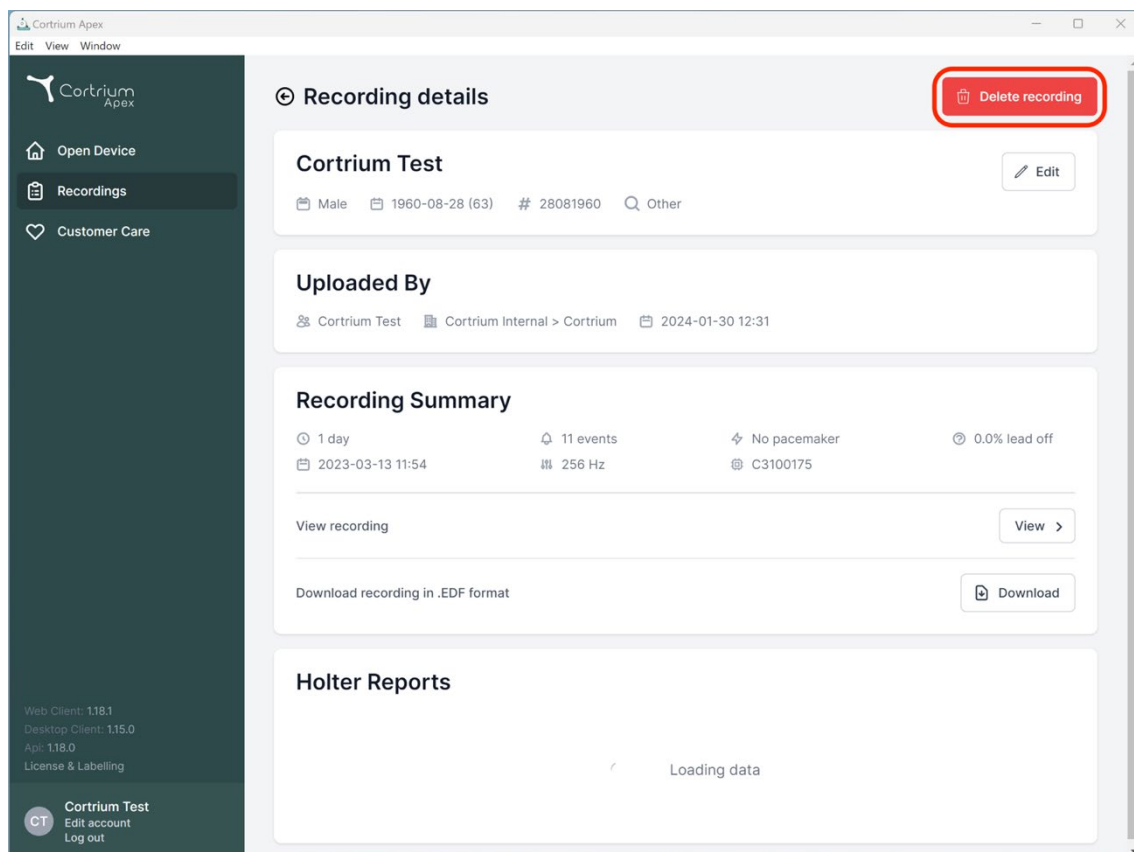
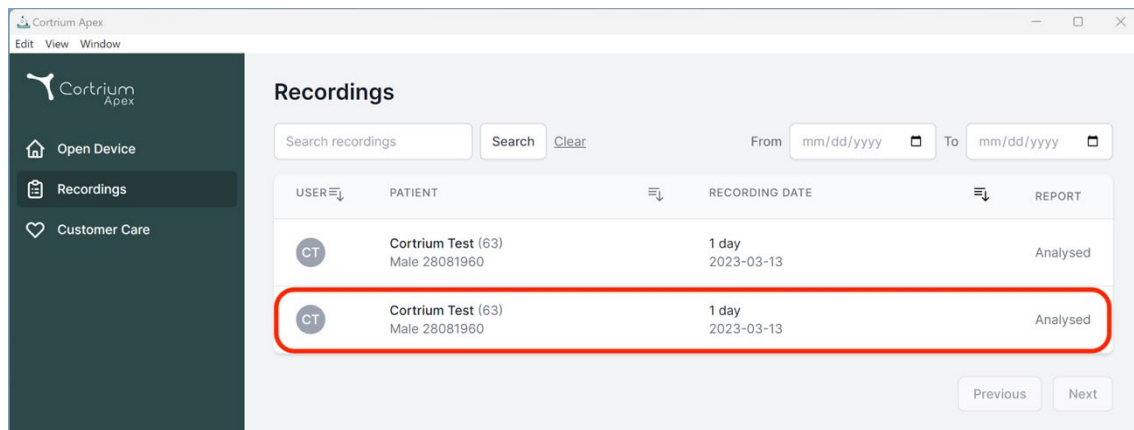
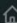
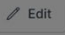
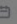
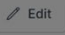
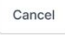
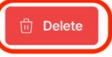
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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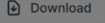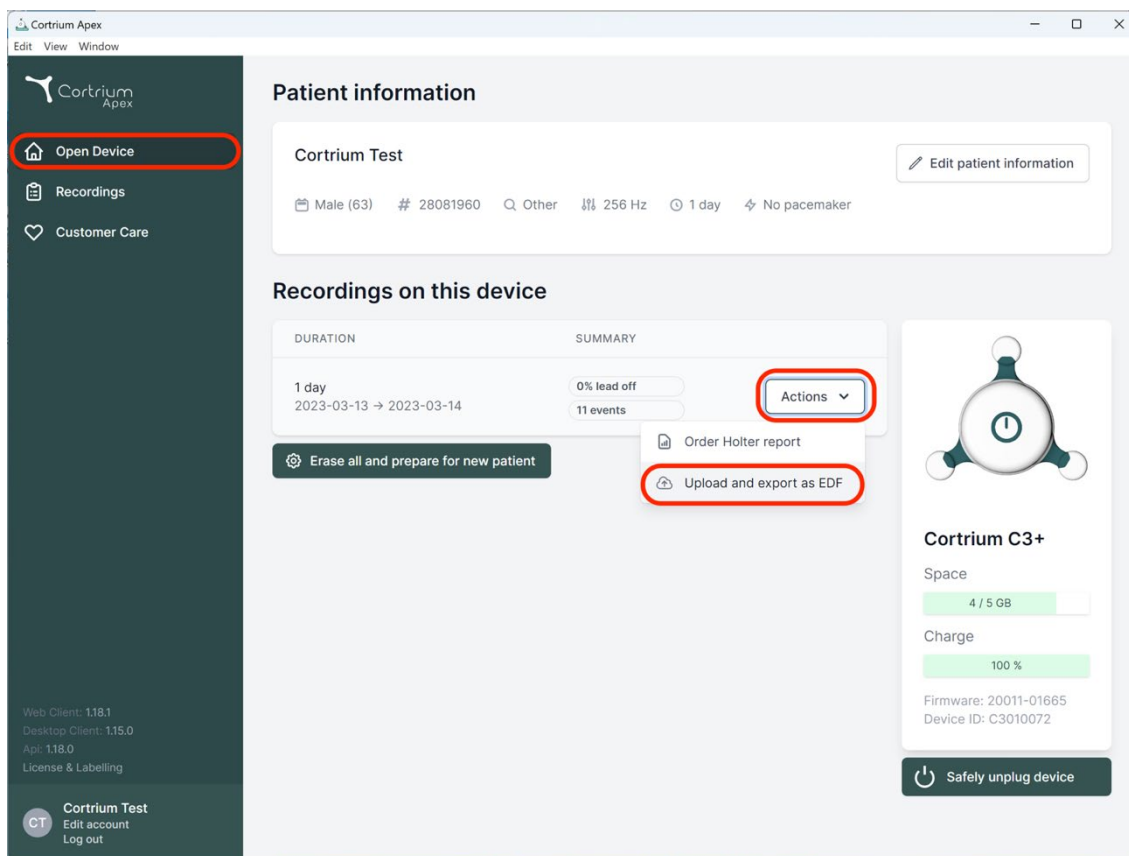
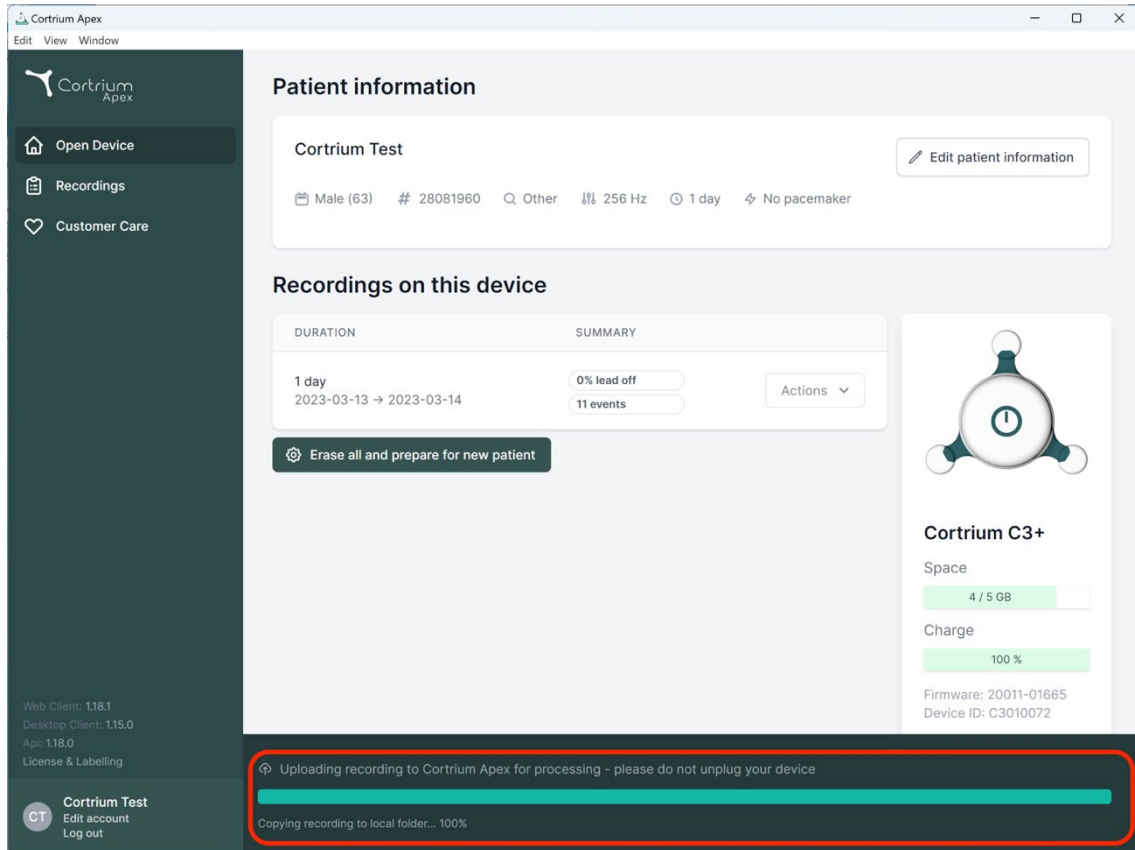
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

| DURATION | SUMMARY | |
|----------------------------------|--------------------------|-------------------------|
| 1 day 2023-03-13 → 2023-03-14 | 0% lead off 11 events | Actions |

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

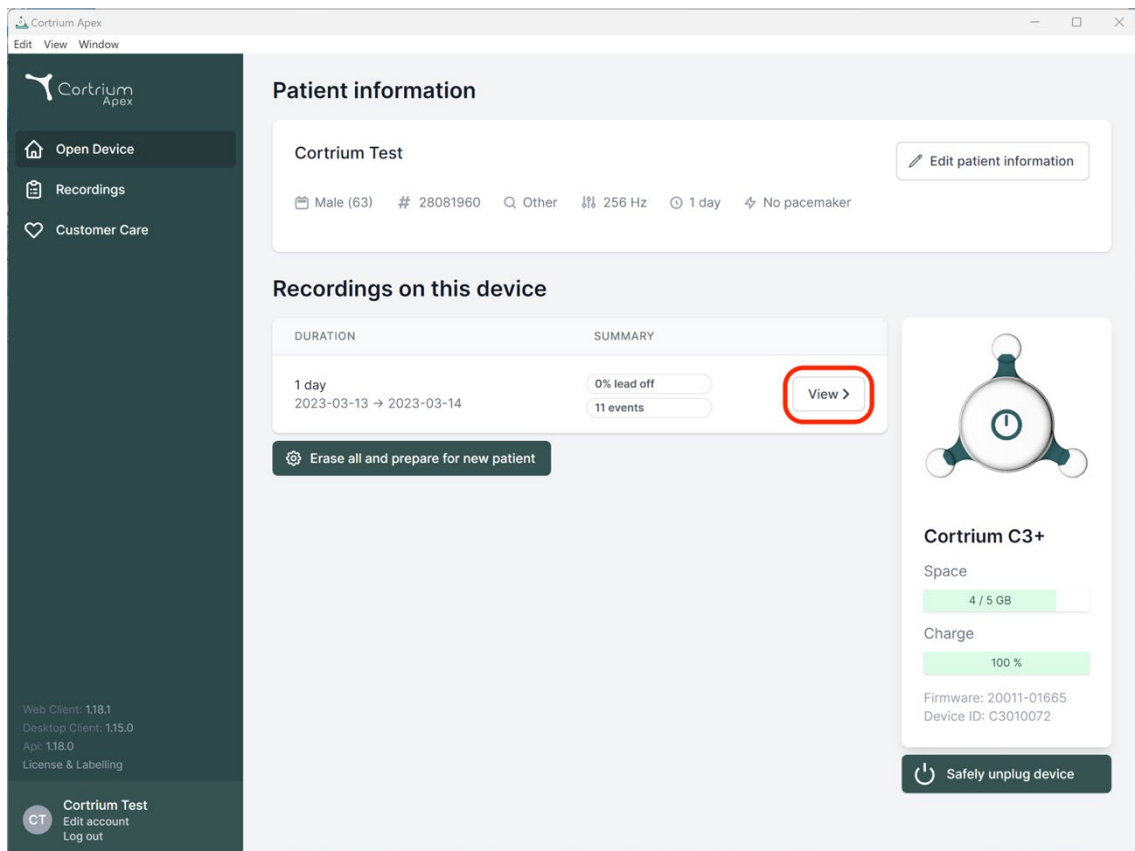
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
Api: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

| DURATION | SUMMARY | |
|----------------------------------|--------------------------|---------------------------|
| 1 day 2023-03-13 → 2023-03-14 | 0% lead off 11 events | View > |

[Erase all and prepare for new patient](#)

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
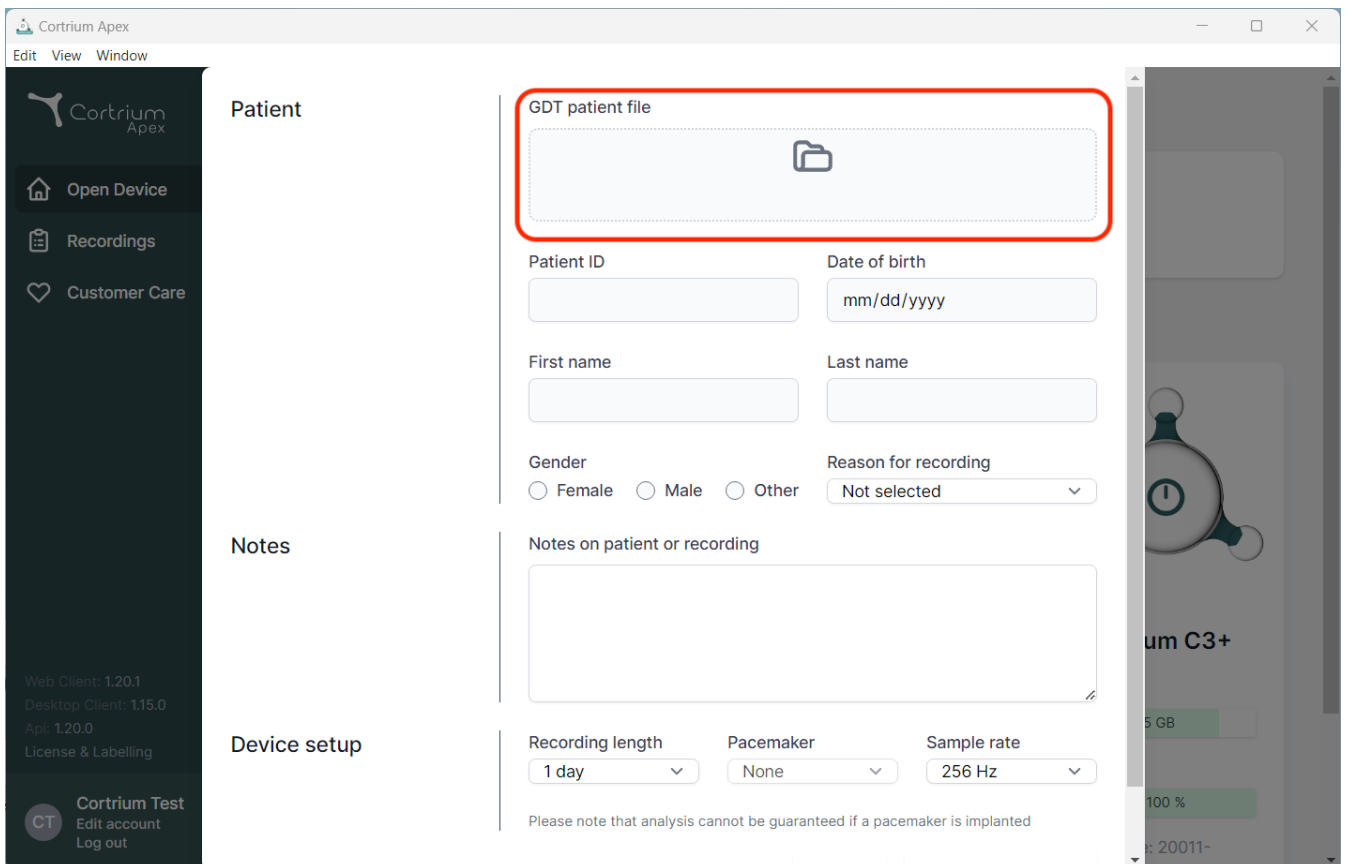
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.20.1, Desktop Client: 1.15.0, App: 1.20.0) and a 'License & Labelling' link. At the bottom of the sidebar is a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area is divided into three sections: 'Patient', 'Notes', and 'Device setup'. The 'Patient' section contains a 'GDT patient file' upload area (highlighted with a red box) and form fields for 'Patient ID', 'Date of birth' (mm/dd/yyyy), 'First name', 'Last name', 'Gender' (Female, Male, Other), and 'Reason for recording' (Not selected). The 'Notes' section has a text area for 'Notes on patient or recording'. The 'Device setup' section includes dropdowns for 'Recording length' (1 day), 'Pacemaker' (None), and 'Sample rate' (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. On the right side of the interface, a partial view of a device screen shows 'um C3+' and a battery level of '100 %'.

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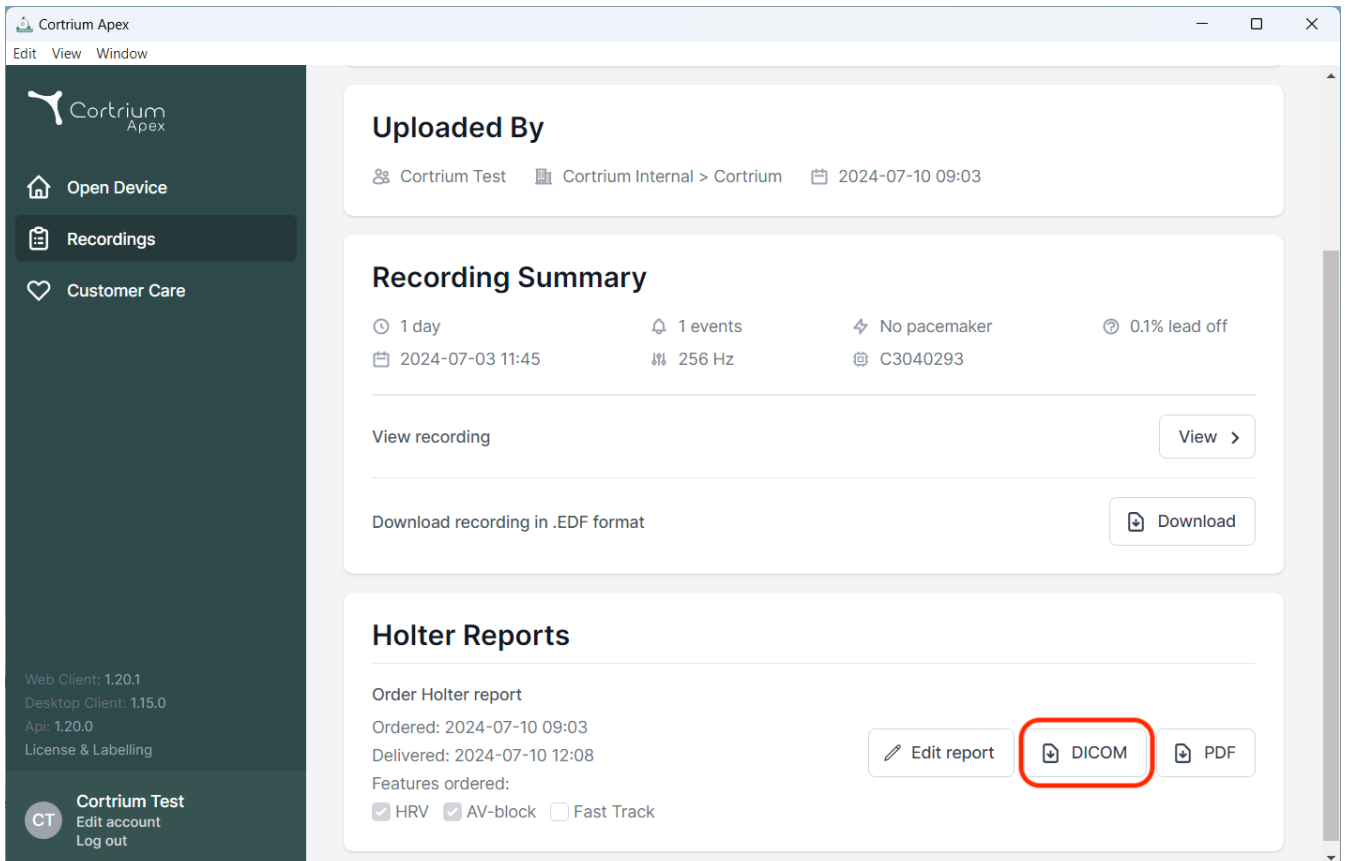The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By', 'Recording Summary', and 'Holter Reports'.

Uploaded By
Cortrium Test | Cortrium Internal > Cortrium | 2024-07-10 09:03

Recording Summary
🕒 1 day | 🔔 1 events | ⚡ No pacemaker | ⚙️ 0.1% lead off
📅 2024-07-03 11:45 | 📡 256 Hz | 🏷️ C3040293

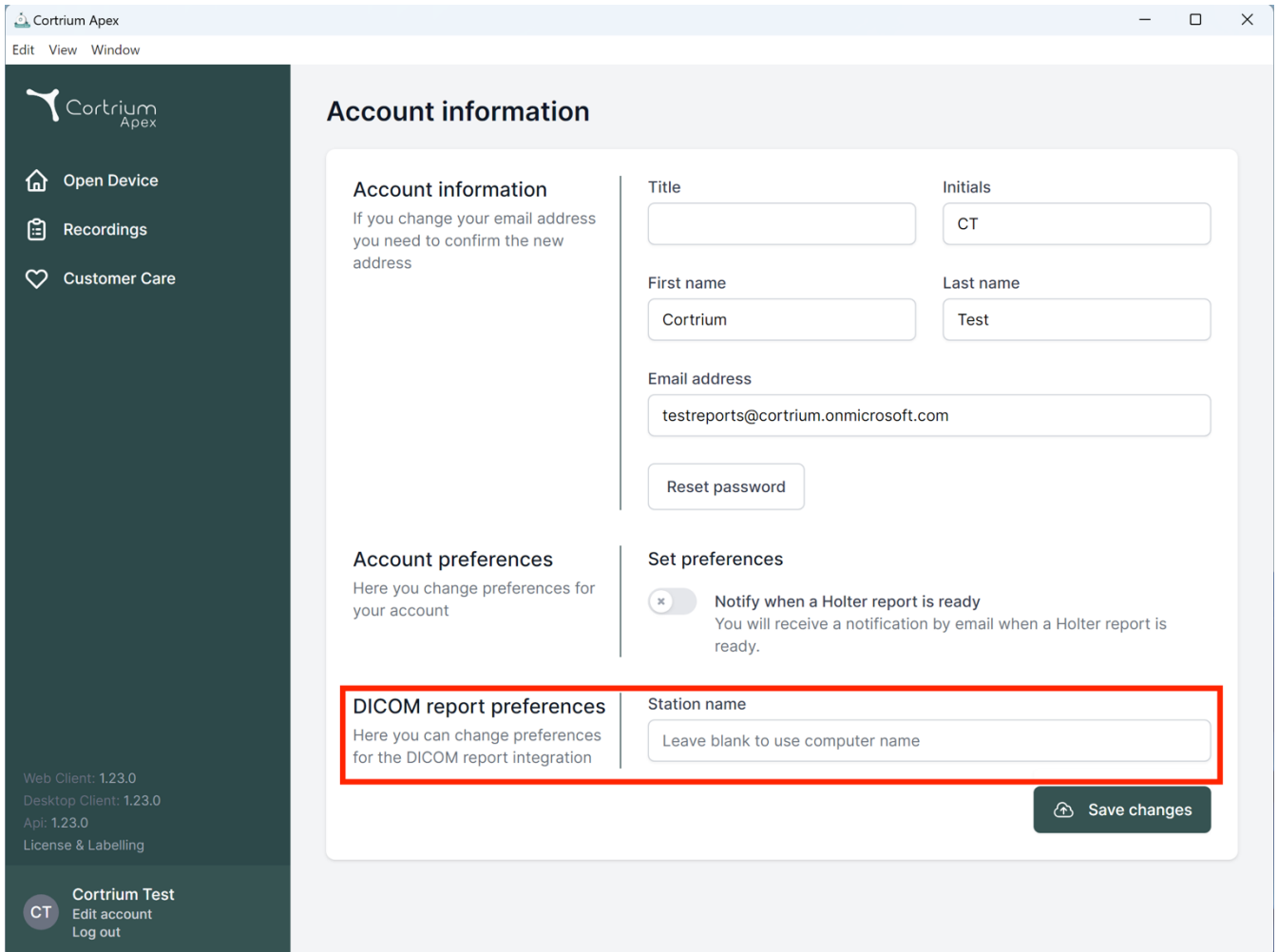
View recording [View >]
Download recording in .EDF format [Download]

Holter Reports
Order Holter report
Ordered: 2024-07-10 09:03
Delivered: 2024-07-10 12:08
Features ordered:
☒ HRV ☒ AV-block ☐ Fast Track

Buttons: [Edit report] [DICOM] [PDF]. The 'DICOM' button is highlighted with a red rectangle.

Footer: Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0, License & Labelling. User: Cortrium Test, Edit account, Log out.

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Account information

If you change your email address you need to confirm the new address

Account information

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

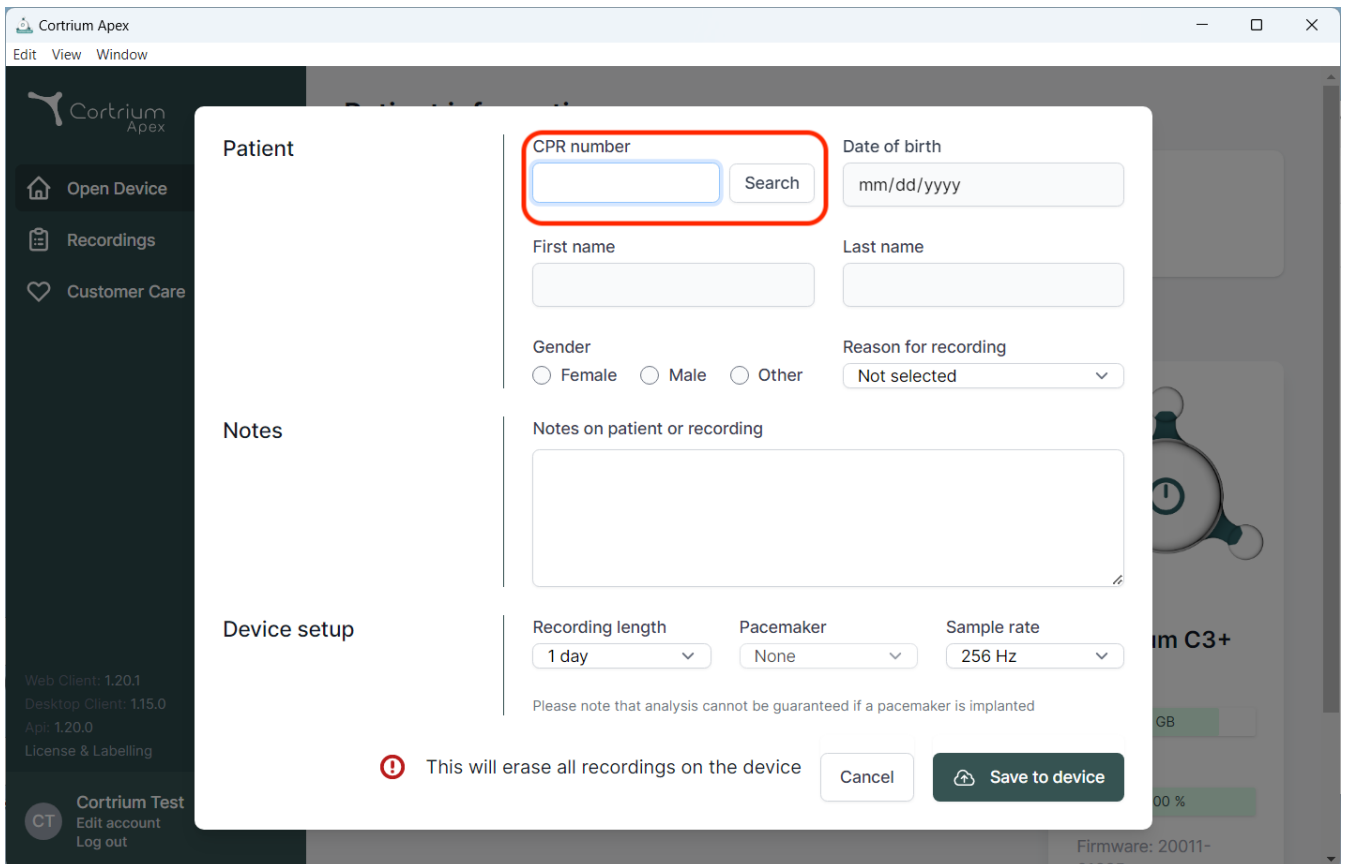
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient information. The "CPR number" field is highlighted with a red box. The form includes fields for "Date of birth", "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), and "Notes on patient or recording". The "Device setup" section includes "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A warning message at the bottom states: "This will erase all recordings on the device". The background shows the Cortrium Apex sidebar with options like "Open Device", "Recordings", and "Customer Care".

Patient

CPR number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

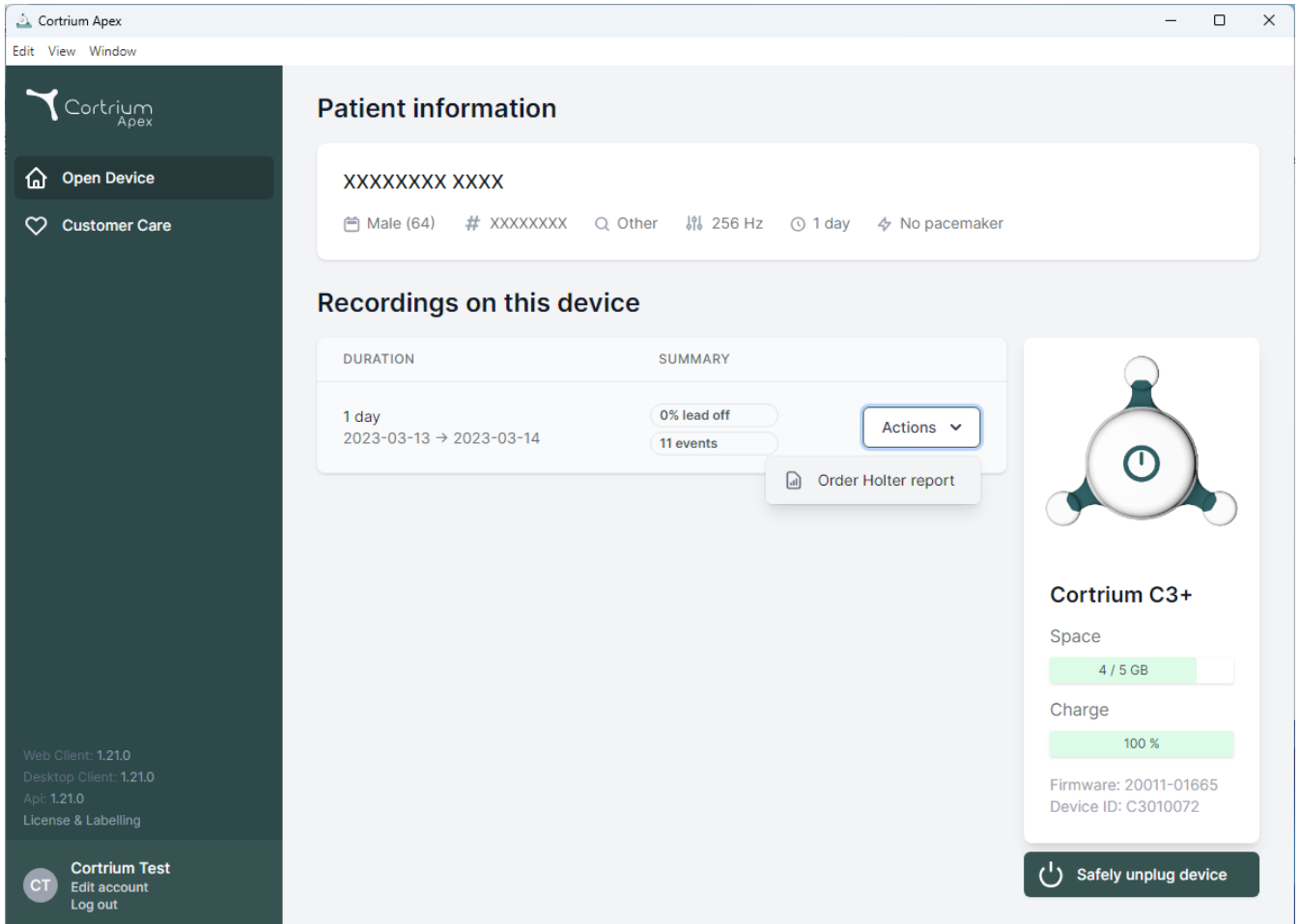
Recording length Pacemaker Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device' (with a house icon), 'Customer Care' (with a heart icon), and a footer section with 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', 'License & Labelling', and a 'Cortrium Test' section with 'Edit account' and 'Log out' links.

The main content area is titled 'Patient information' and shows a patient ID 'XXXXXXXX XXXX'. Below this, patient details are listed: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'. The 'Recordings on this device' section contains a table with two columns: 'DURATION' and 'SUMMARY'. The table has one entry for '1 day' (2023-03-13 → 2023-03-14) with '0% lead off' and '11 events'. An 'Actions' dropdown menu is visible next to the recording, with an option to 'Order Holter report'. On the right, a device status card for 'Cortrium C3+' shows 'Space' (4 / 5 GB), 'Charge' (100 %), 'Firmware: 20011-01665', and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom right.

| DURATION | SUMMARY |
|----------------------------------|--------------------------|
| 1 day 2023-03-13 → 2023-03-14 | 0% lead off 11 events |

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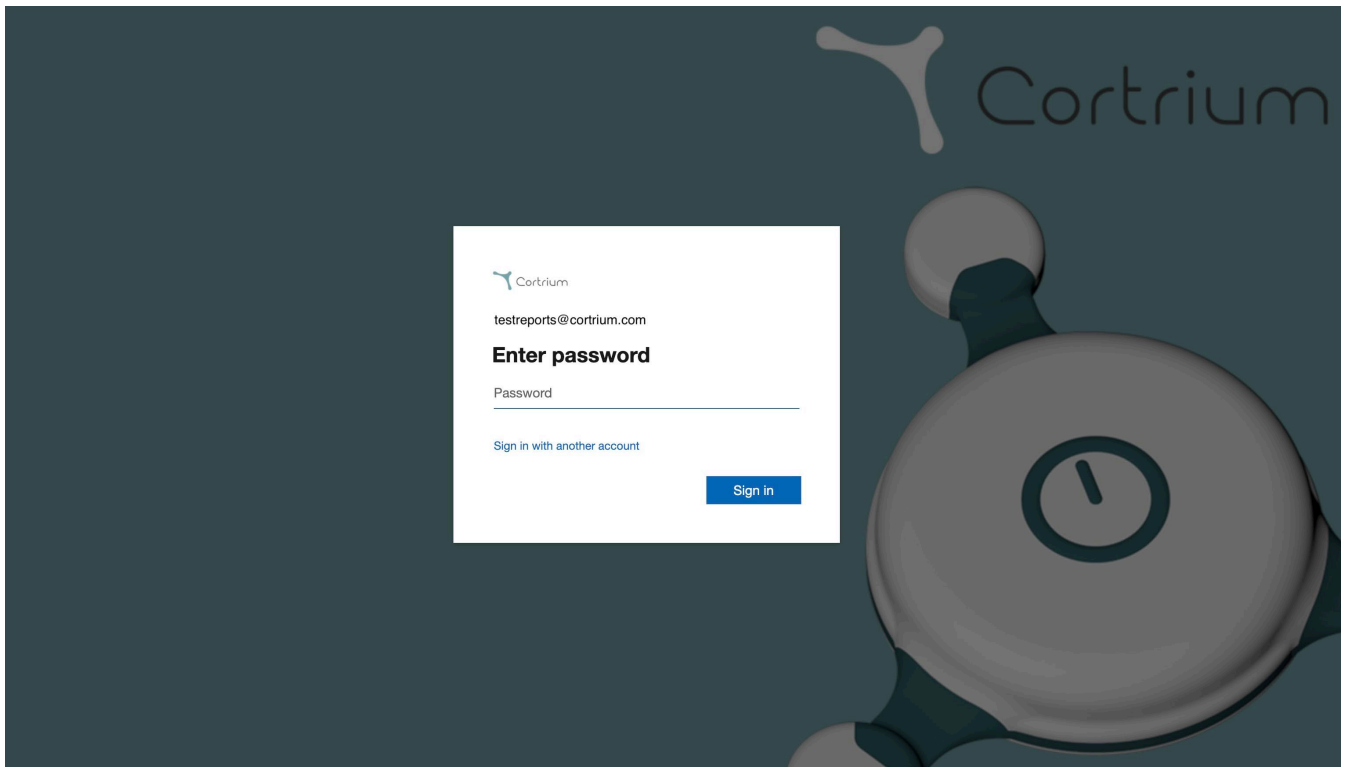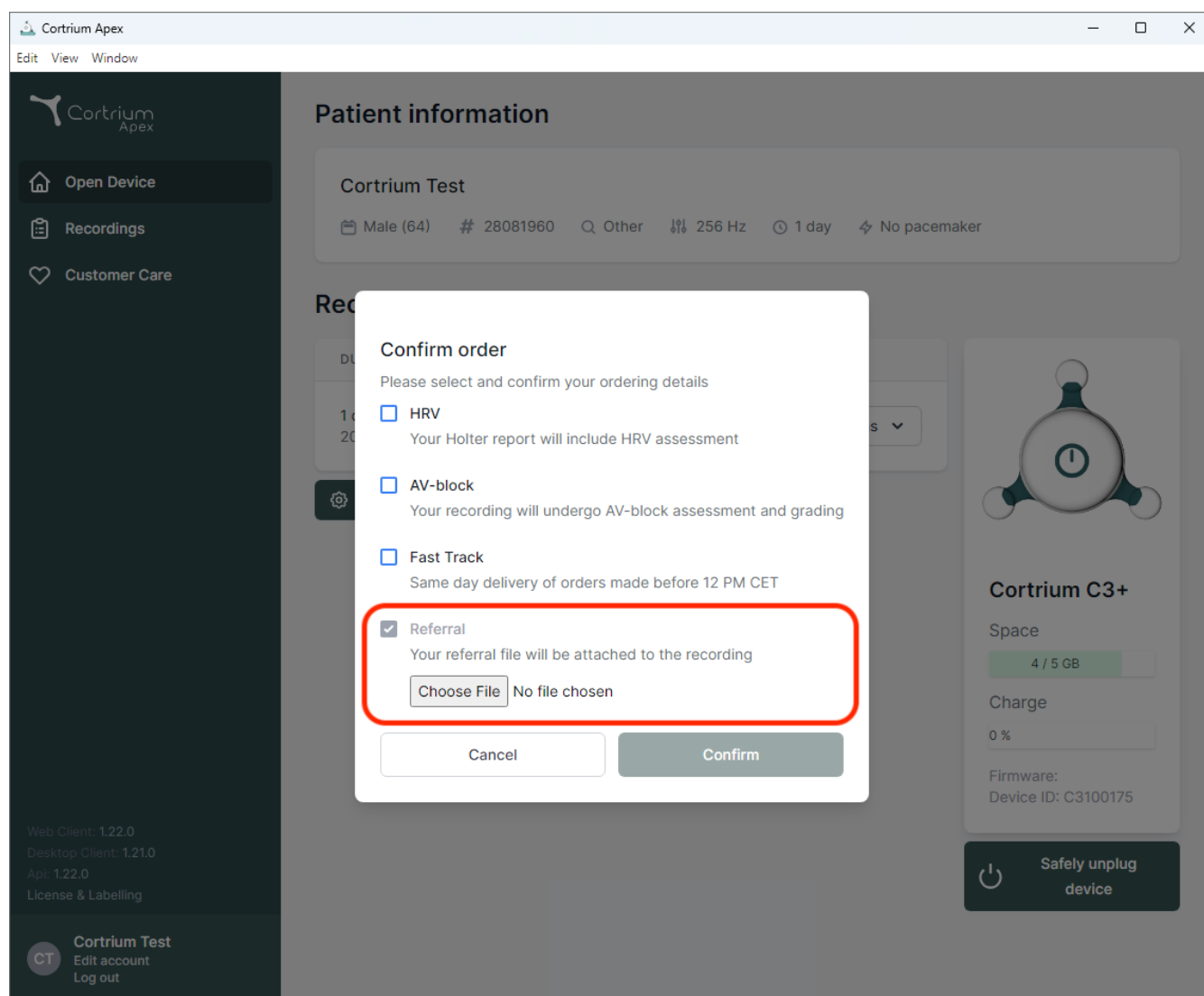
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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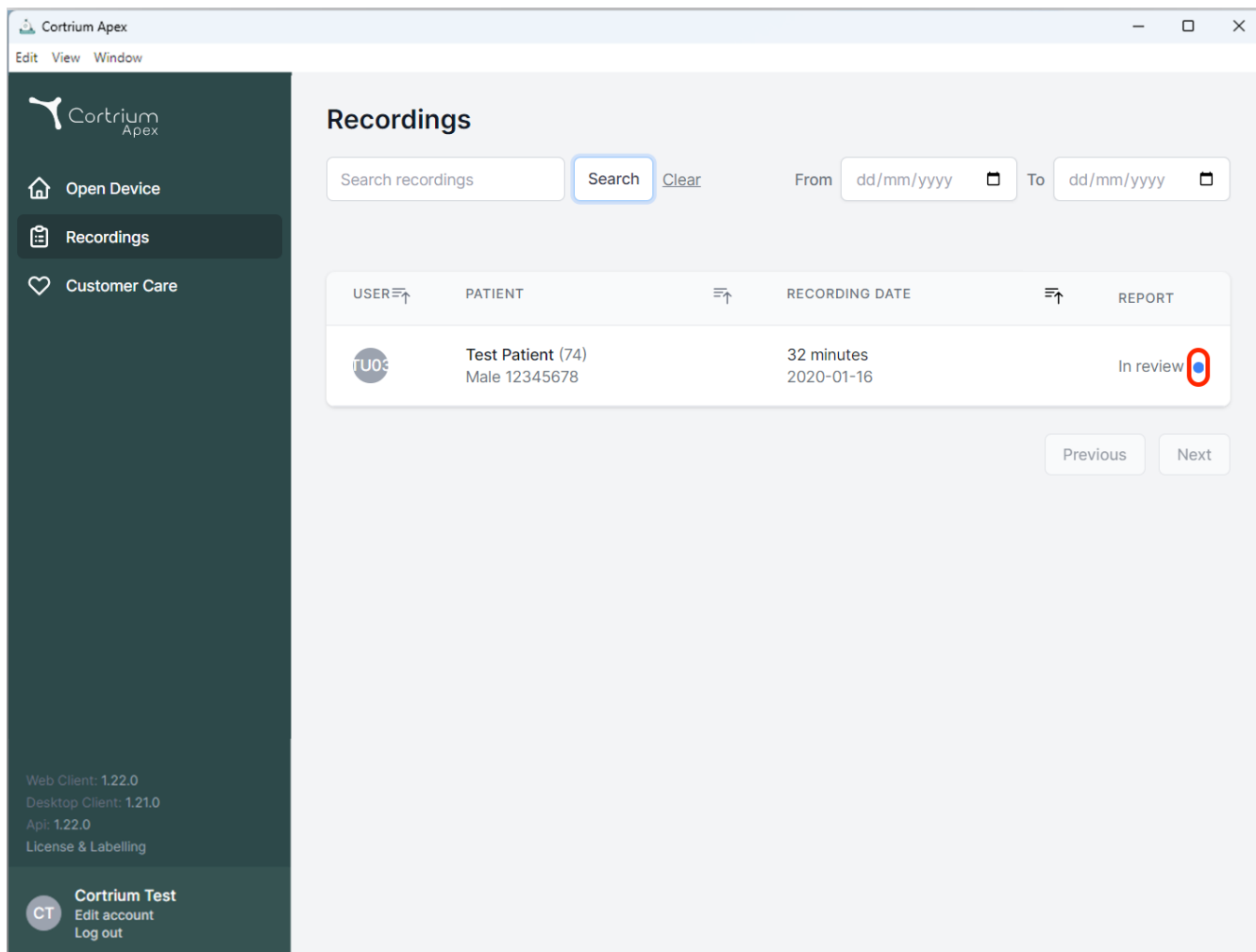
Recordings

Search recordings [Clear](#) From To


| USER | PATIENT | RECORDING DATE | REPORT |
|------|------------------------------------|--------------------------|---|
| TU03 | Test Patient (74) Male 12345678 | 32 minutes 2020-01-16 | In review  |

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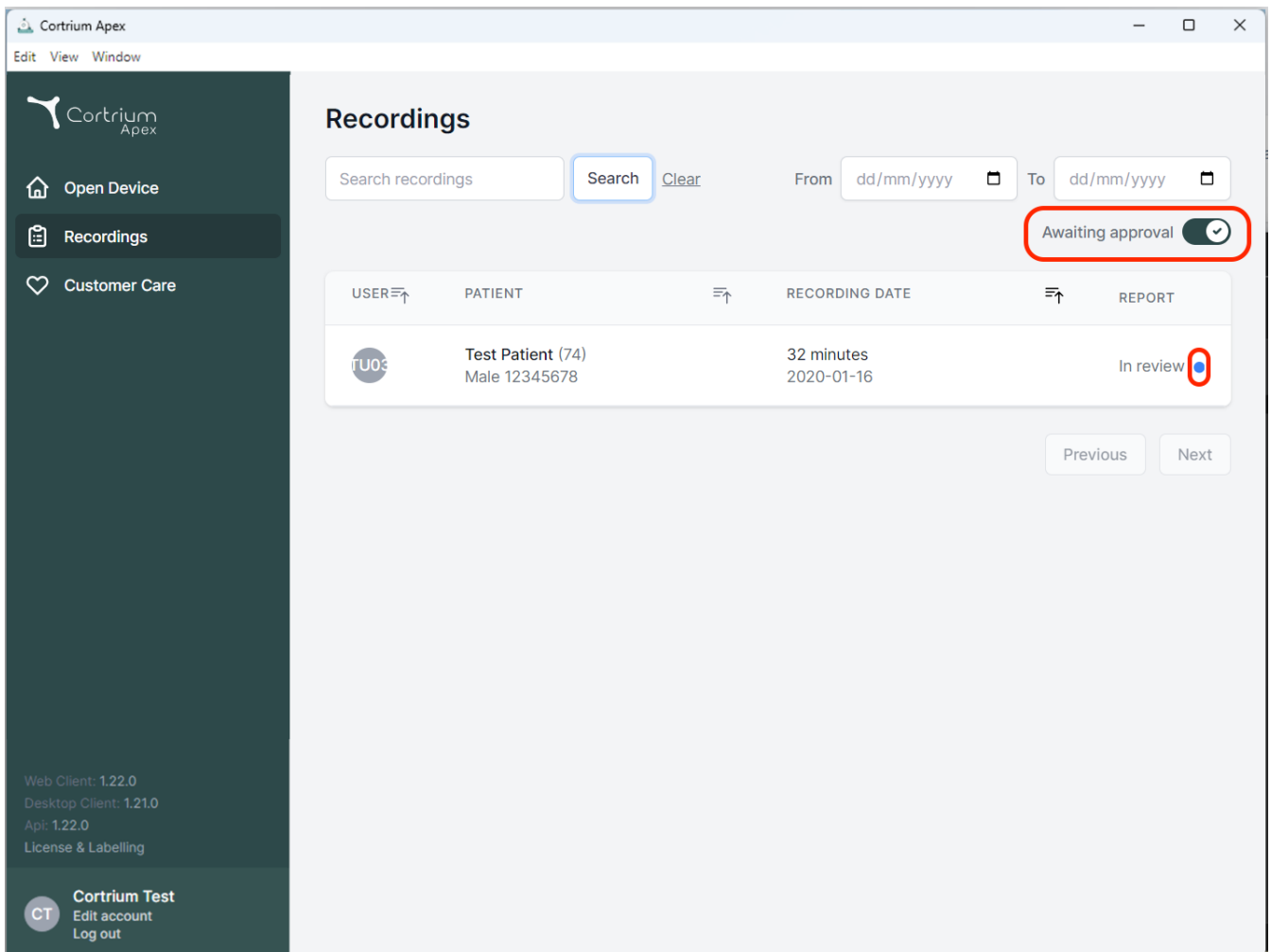
From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒

| USER | PATIENT | RECORDING DATE | REPORT |
|------|------------------------------------|--------------------------|-----------|
| TU03 | Test Patient (74) Male 12345678 | 32 minutes 2020-01-16 | In review |

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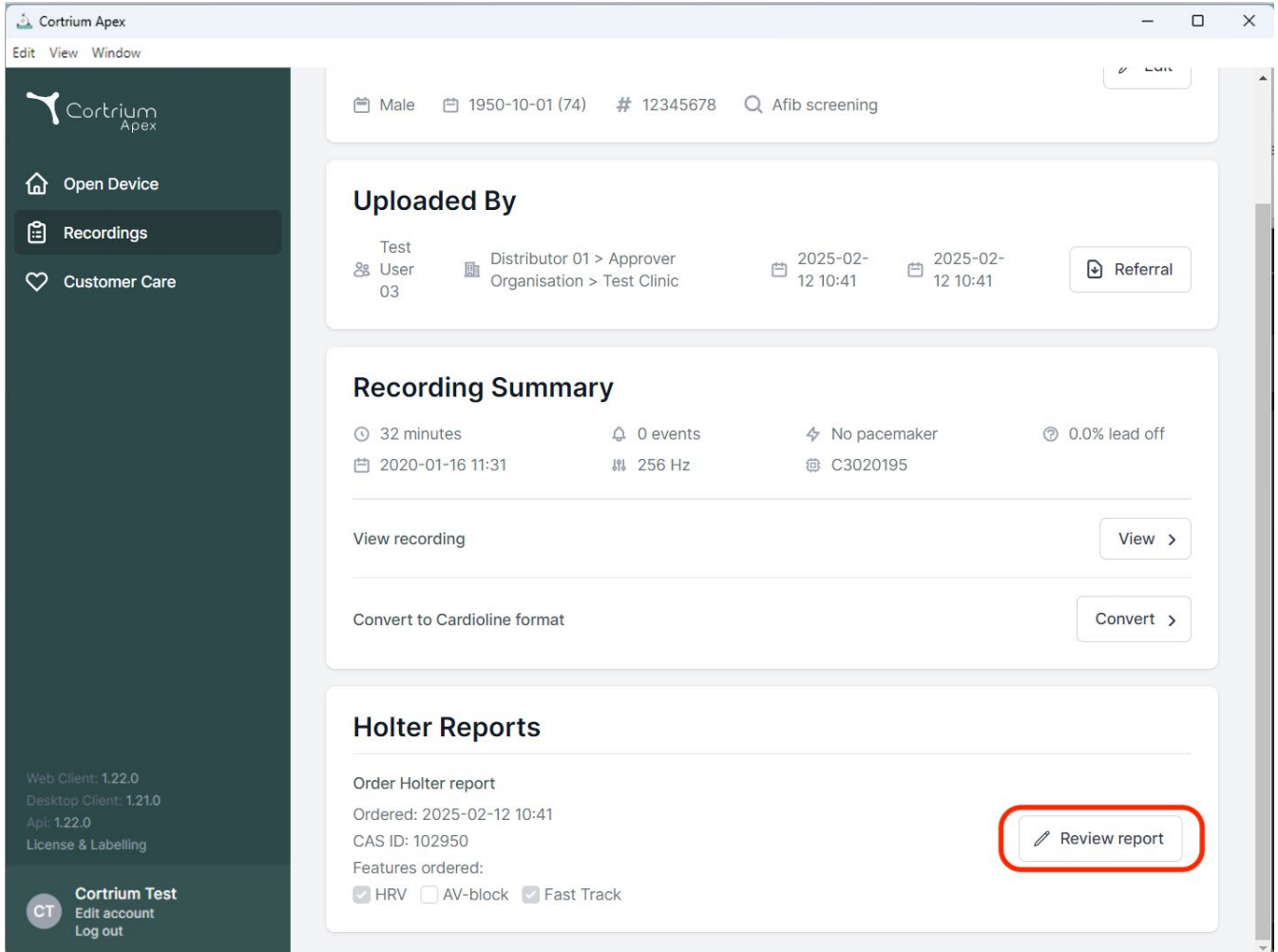The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' menus. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area shows patient details: Male, 1950-10-01 (74), # 12345678, and Afib screening. Below this, the 'Uploaded By' section shows 'Test User 03' and 'Distributor 01 > Approver Organisation > Test Clinic' with timestamps '2025-02-12 10:41' and a 'Referral' button. The 'Recording Summary' section displays '32 minutes', '0 events', 'No pacemaker', and '0.0% lead off', along with a date '2020-01-16 11:31', '256 Hz', and 'C3020195'. It includes buttons for 'View recording' and 'Convert to Cardioline format'. The 'Holter Reports' section shows 'Order Holter report', 'Ordered: 2025-02-12 10:41', 'CAS ID: 102950', and 'Features ordered: HRV, AV-block, Fast Track'. A 'Review report' button is highlighted with a red rectangle.

Uploaded By

Test User 03 Distributor 01 > Approver Organisation > Test Clinic 2025-02-12 10:41 2025-02-12 10:41 Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
 2020-01-16 11:31 256 Hz C3020195

View recording View >

Convert to Cardioline format Convert >

Holter Reports

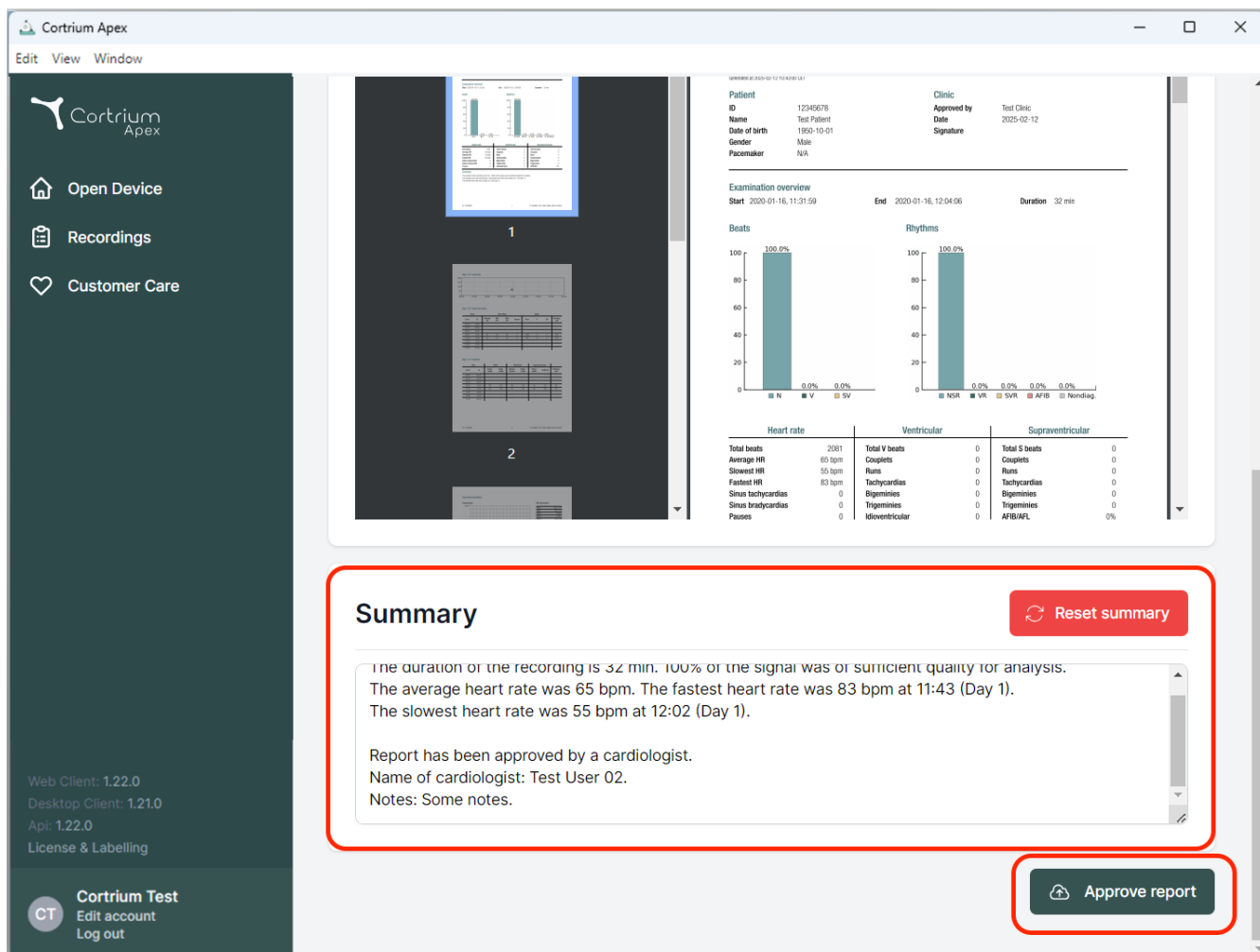
Order Holter report
 Ordered: 2025-02-12 10:41
 CAS ID: 102950
 Features ordered:
☒ HRV ☐ AV-block ☒ Fast Track

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Cortrium Apex
Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary Reset summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report

Examination overview
Start: 2020-01-16, 11:31:59 End: 2020-01-16, 12:04:06 Duration: 32 min

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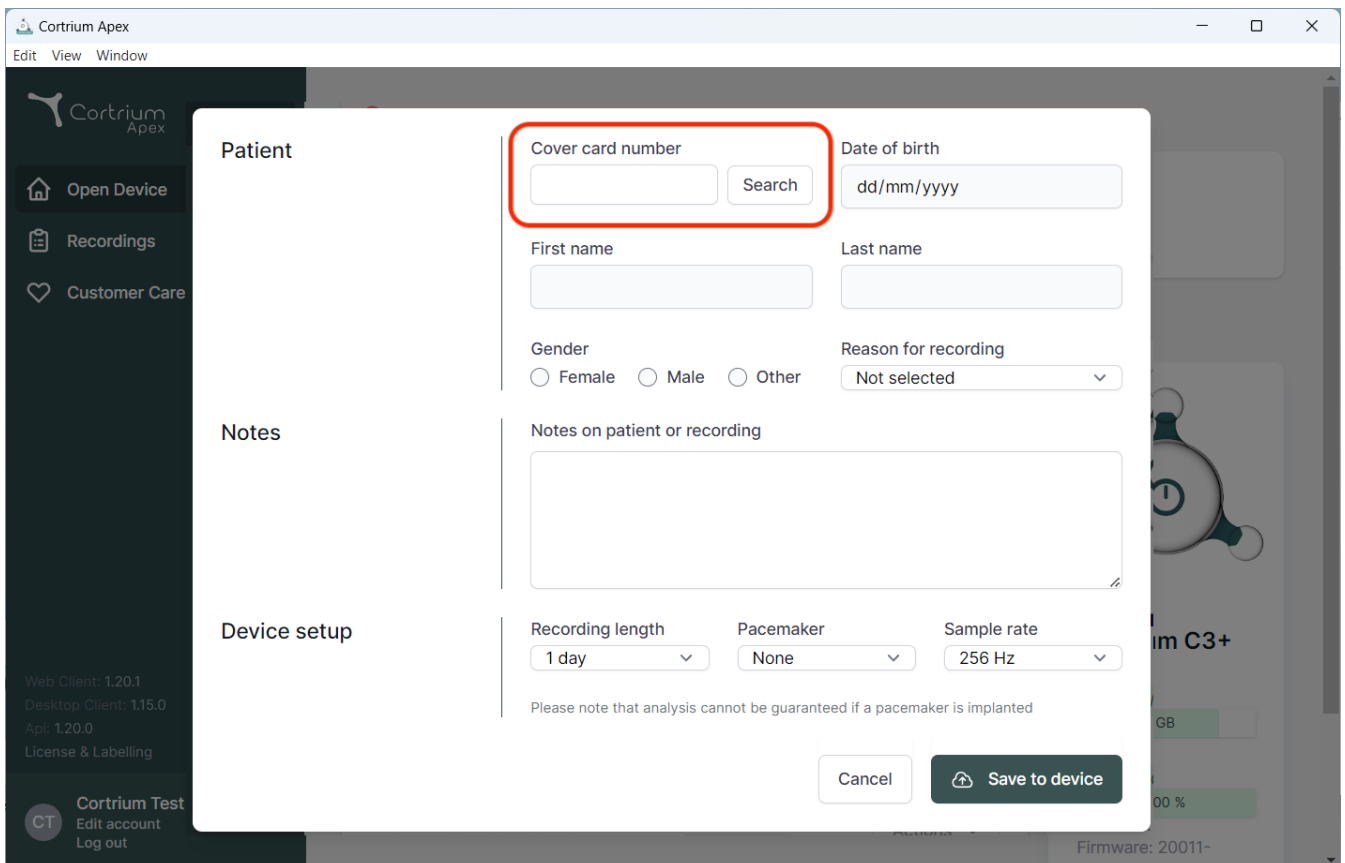
The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, showing a form for patient information. The "Cover card number" field is highlighted with a red box, and a "Search" button is next to it. The form includes fields for "Date of birth" (dd/mm/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), "Notes on patient or recording", "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A "Cancel" button and a "Save to device" button are at the bottom right. The background shows the Cortrium Apex dashboard with a sidebar menu and a main content area.

Patient

Cover card number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length Pacemaker Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Cancel

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dinamarca
E-mail: info@cortrium.com Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
46 / 46