Cortrium Apex 1.27

Cortrium Apex 1.27

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
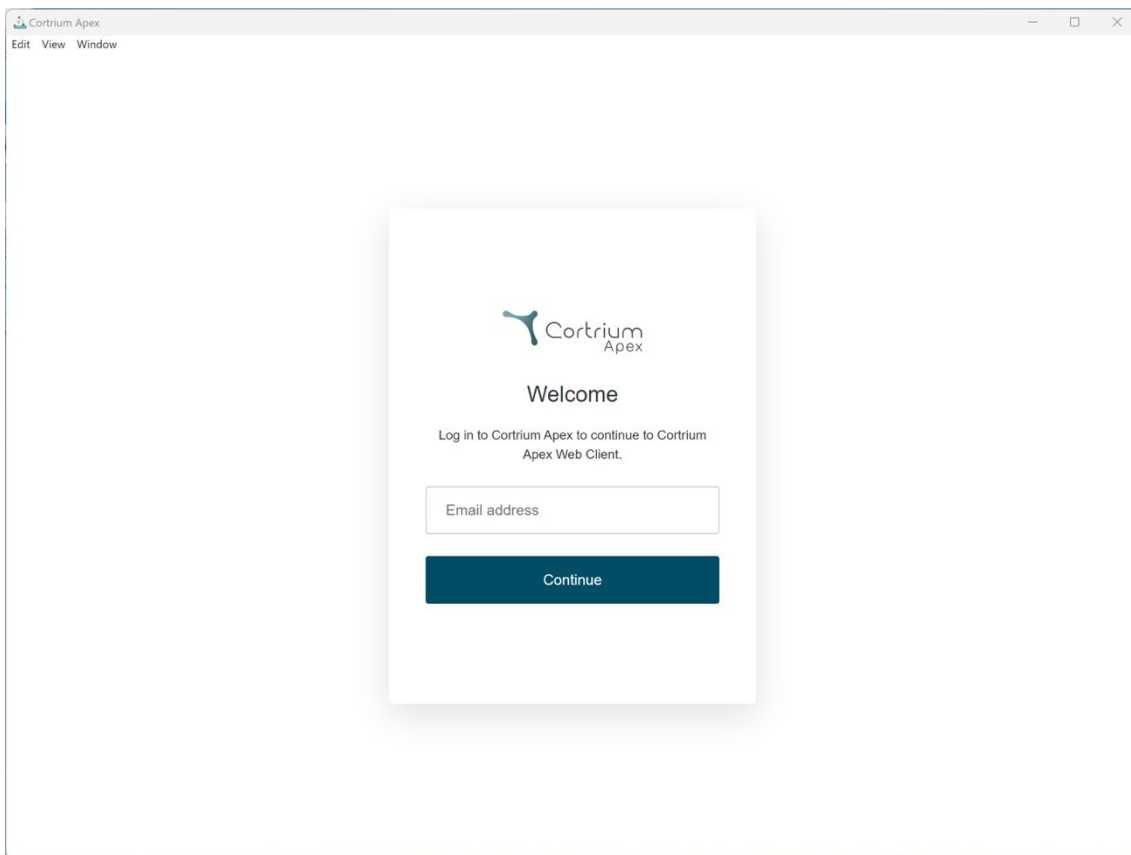
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". The main content area displays a login form centered on the page. The form has a white background and a subtle drop shadow. At the top of the form is the Cortrium Apex logo. Below the logo is the heading "Welcome". Underneath the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." followed by a text input field labeled "Email address". At the bottom of the form is a dark blue button with the text "Continue" in white.

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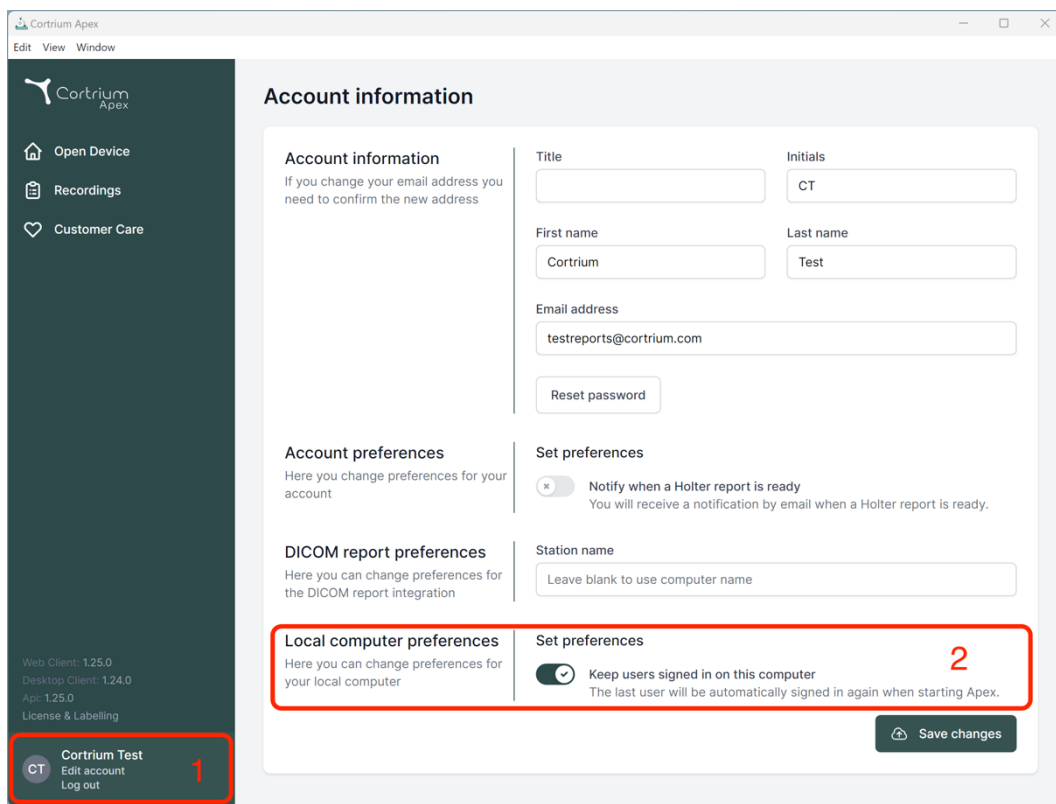
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client, Desktop Client, and API, along with a 'License & Labelling' link. At the bottom of the sidebar is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out', marked with a red '1'. The main content area is titled 'Account information' and contains several sections: 'Account information' with fields for Title, Initials, First name, Last name, and Email address; 'Account preferences' for changing account settings; 'DICOM report preferences' for integration settings; and 'Local computer preferences' for local device settings. The 'Local computer preferences' section includes a 'Set preferences' area with a toggle for 'Keep users signed in on this computer', which is currently turned on and marked with a red '2'. A 'Save changes' button is located at the bottom right of the main content area.

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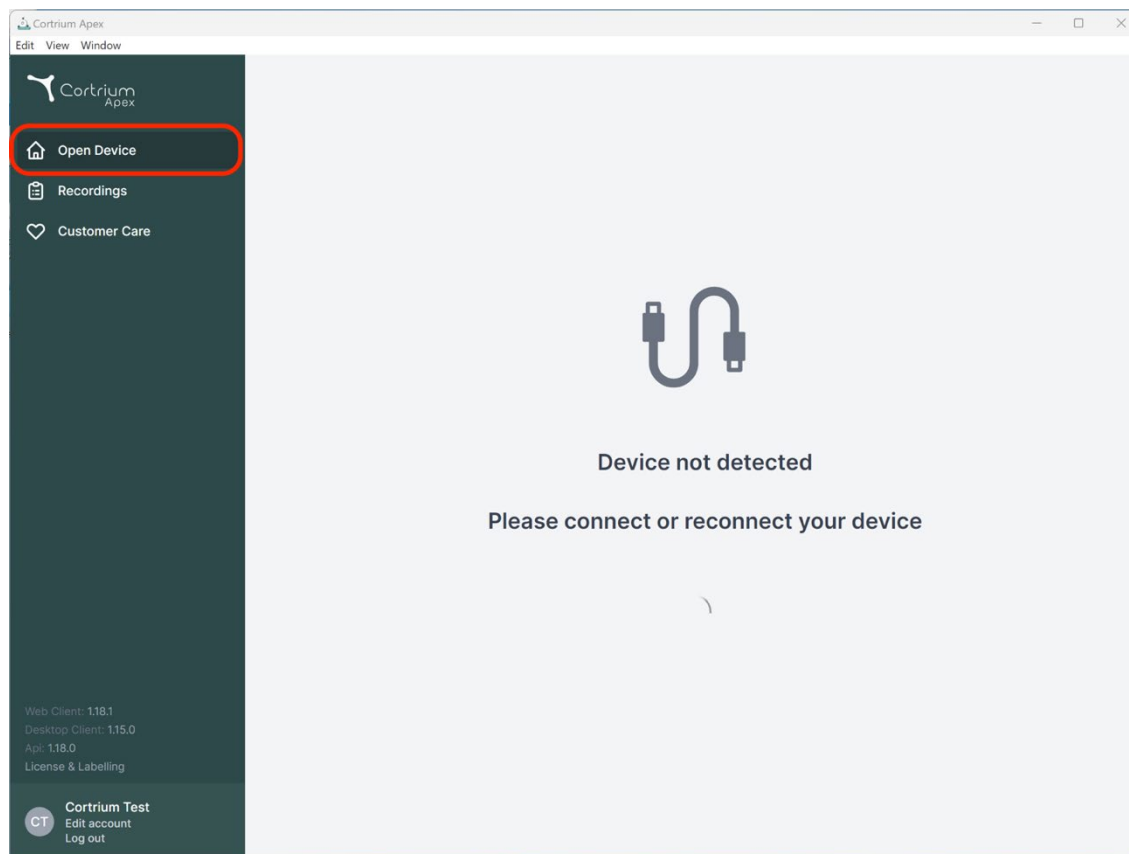
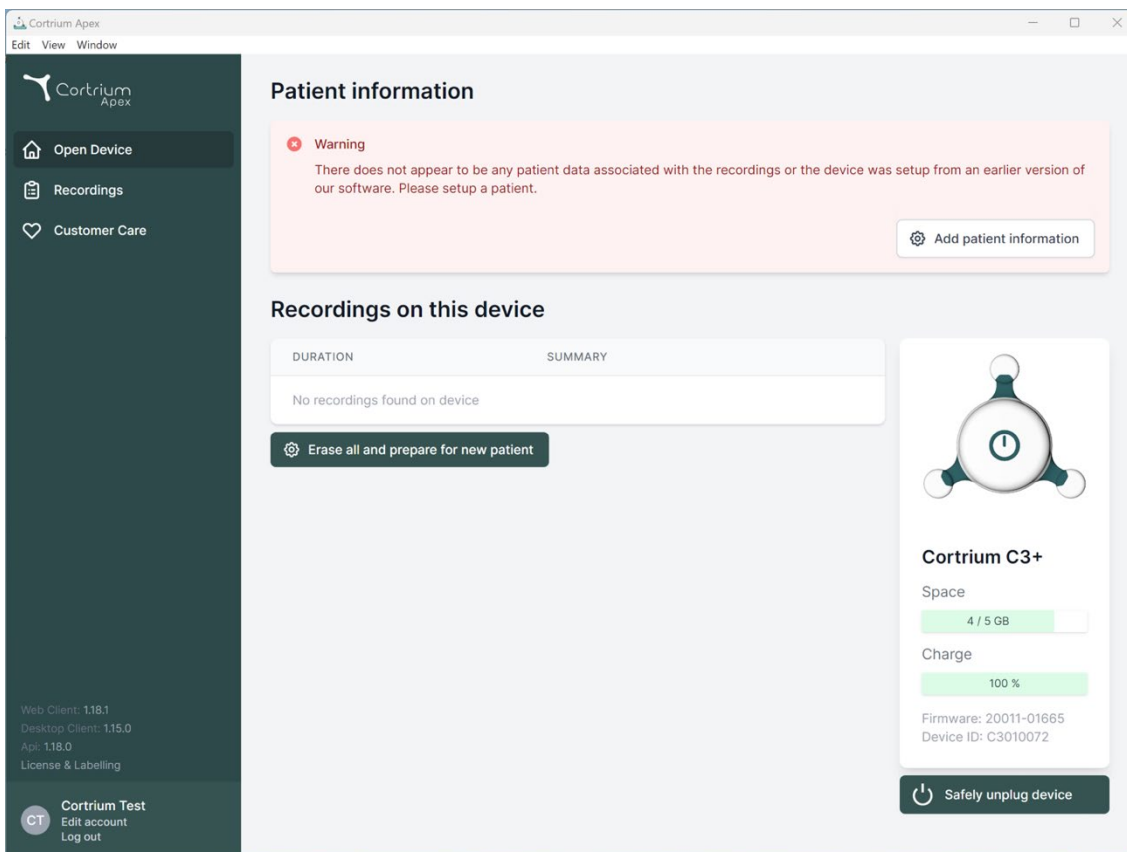
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' links. The main content area is titled 'Patient information' and features a red warning box stating: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is the 'Recordings on this device' section, which shows a table with columns 'DURATION' and 'SUMMARY'. The table is empty, displaying 'No recordings found on device', and includes an 'Erase all and prepare for new patient' button. On the right side of the main area, there is a device status card for 'Cortrium C3+'. It shows a battery icon, 'Space' usage at '4 / 5 GB', 'Charge' at '100 %', and technical details: 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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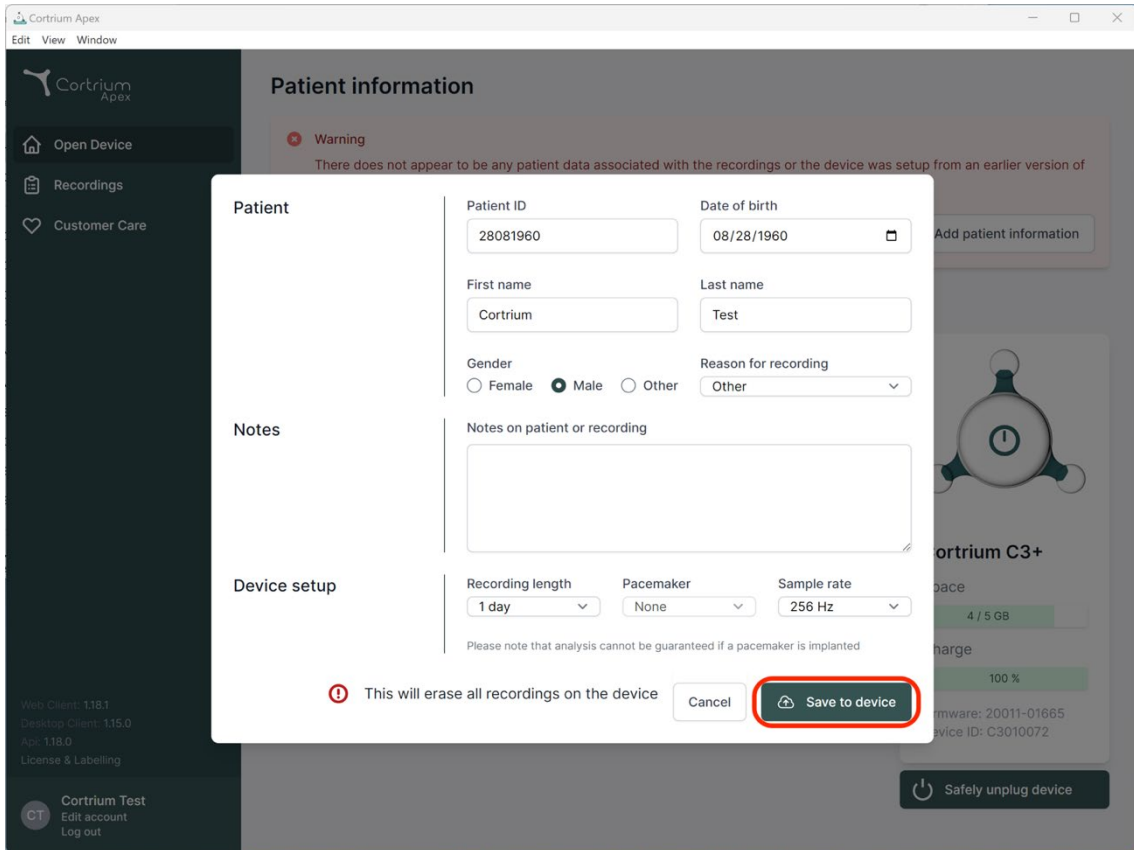Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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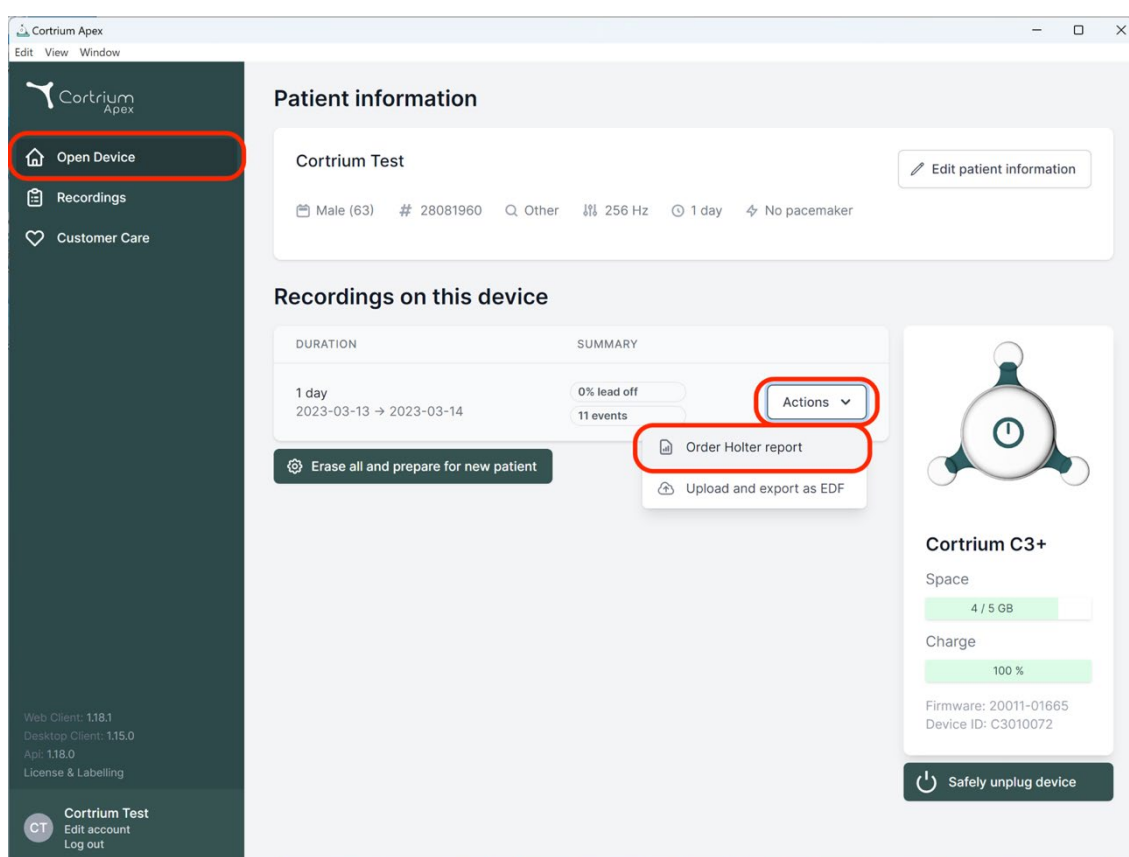
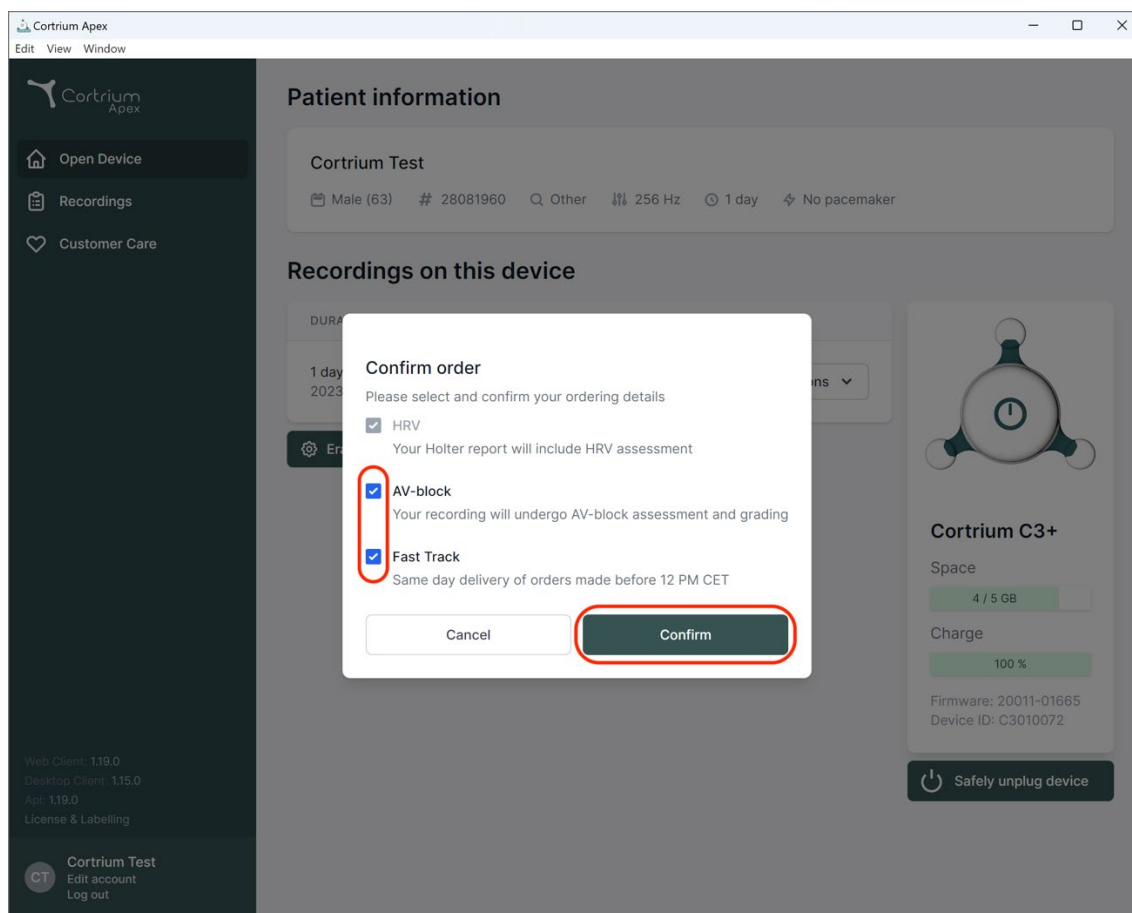
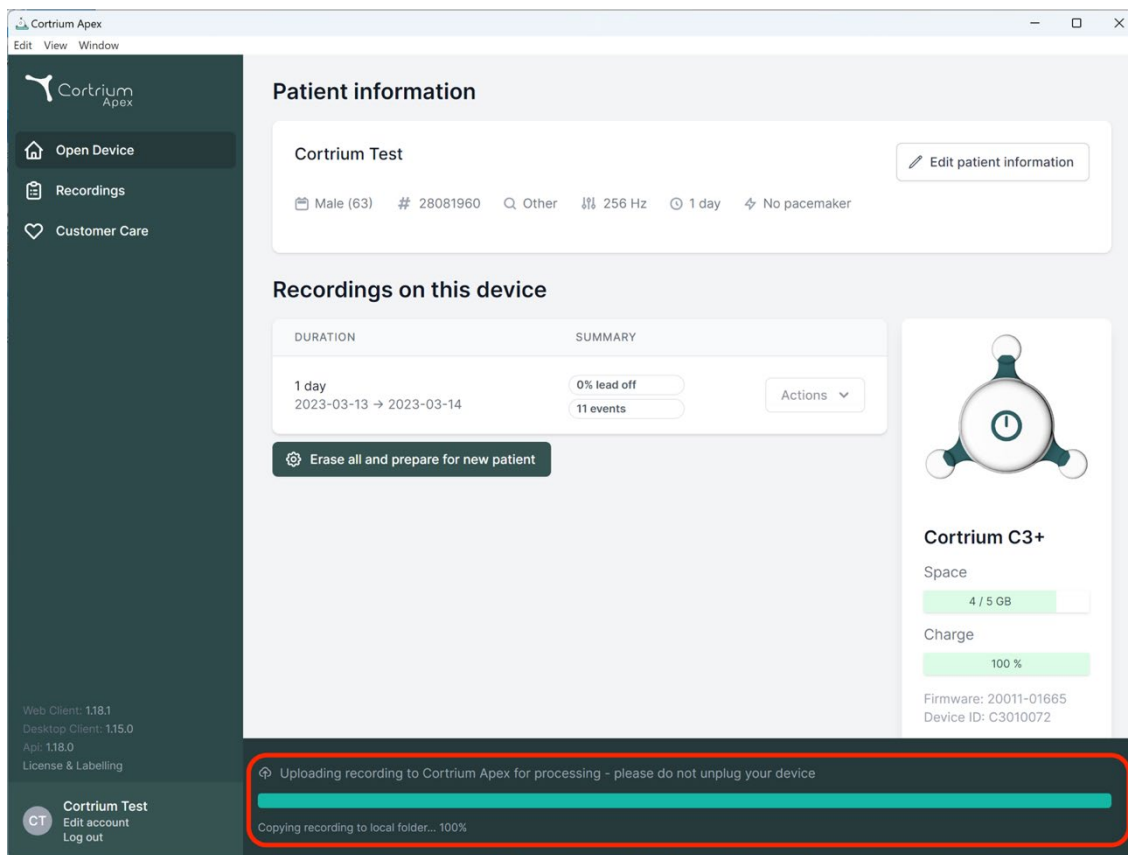
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and other parameters. Below this, the 'Recordings on this device' section shows a recording for '1 day' (2023-03-13 to 2023-03-14) with a summary of '0% lead off' and '11 events'. A button 'Erase all and prepare for new patient' is visible. On the right, the 'Cortrium C3+' device status is shown, including space (4 / 5 GB) and charge (100%). At the bottom, a red-bordered box highlights a progress bar and status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. The bottom left corner shows the user's account information: 'Cortrium Test', 'Edit account', and 'Log out'.

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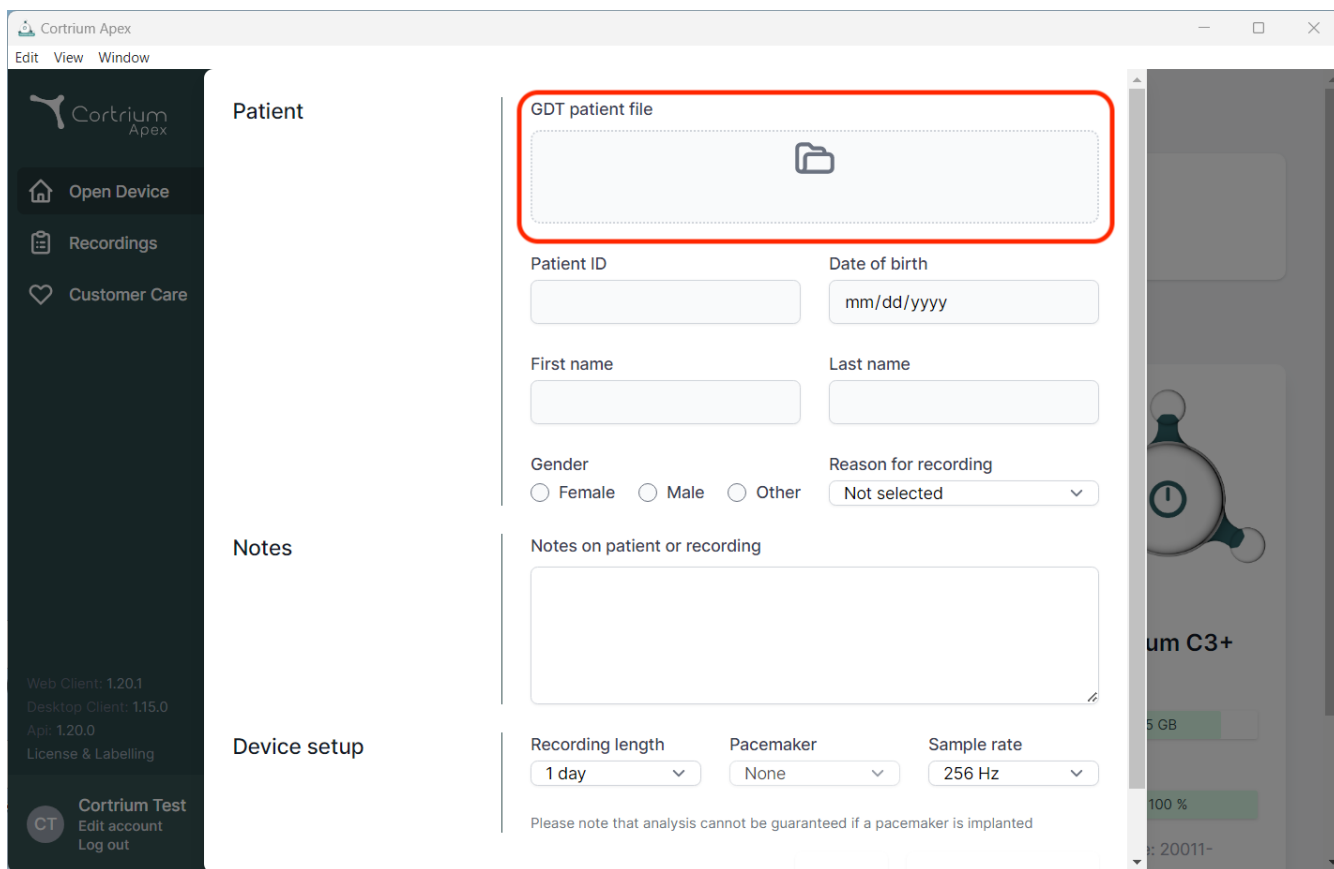
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings", and "Customer Care". Below these are version details: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". At the bottom of the sidebar is a "Cortrium Test" section with "Edit account" and "Log out" options.

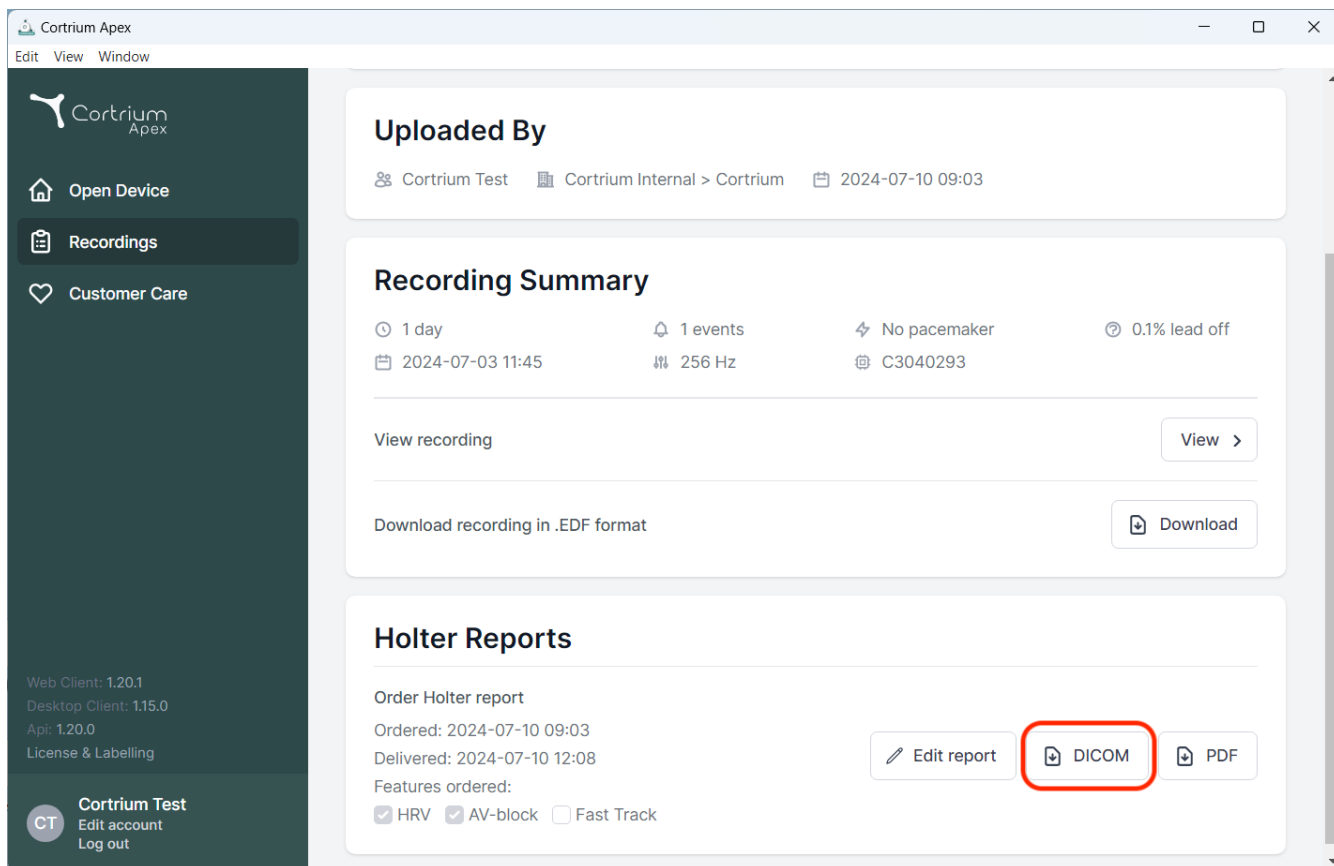
The main content area is divided into three sections:

- Patient:** This section contains a "GDT patient file" upload area, which is highlighted with a red rectangle. Below this are input fields for "Patient ID", "Date of birth" (with a "mm/dd/yyyy" placeholder), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected").
- Notes:** This section features a text area labeled "Notes on patient or recording".
- Device setup:** This section includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

On the right side of the interface, a partial view of a device screen is visible, showing a "Cortrium C3+" device with a battery level indicator at 100% and a date "2011-".

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By' (showing 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'), 'Recording Summary' (displaying device details like '1 day', '1 events', 'No pacemaker', and '0.1% lead off'), and 'Holter Reports'. In the 'Holter Reports' section, under 'Order Holter report', there are buttons for 'Edit report', 'DICOM' (which is highlighted with a red rectangle), and 'PDF'. Below these buttons, the 'Features ordered' section shows 'HRV' and 'AV-block' as selected options, and 'Fast Track' as an unselected option.

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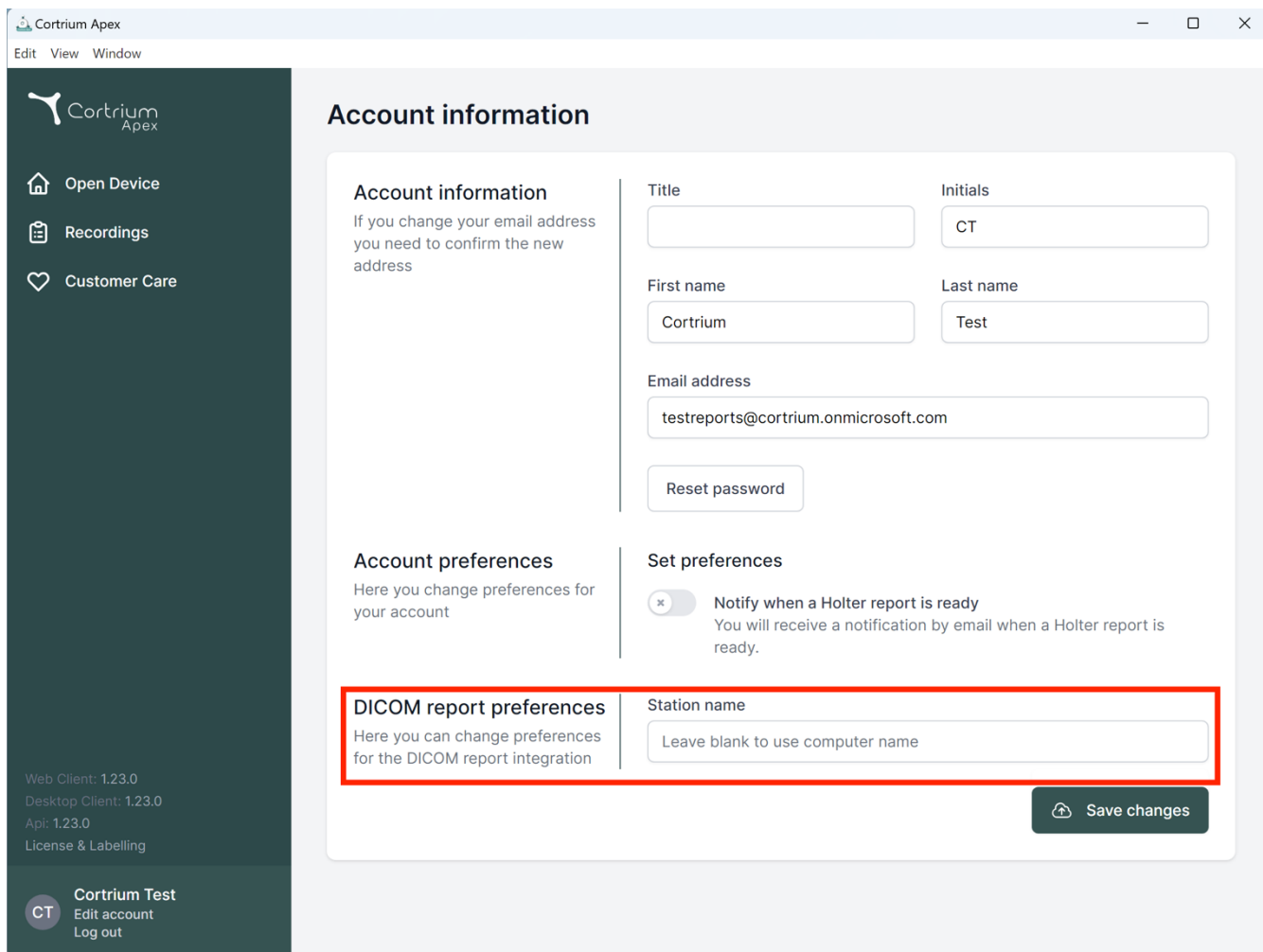Account information

If you change your email address you need to confirm the new address

Account information

Title:

Initials:

First name:

Last name:

Email address:

[Reset password](#)

Account preferences

Here you change preferences for your account

Set preferences

☒ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

Station name:

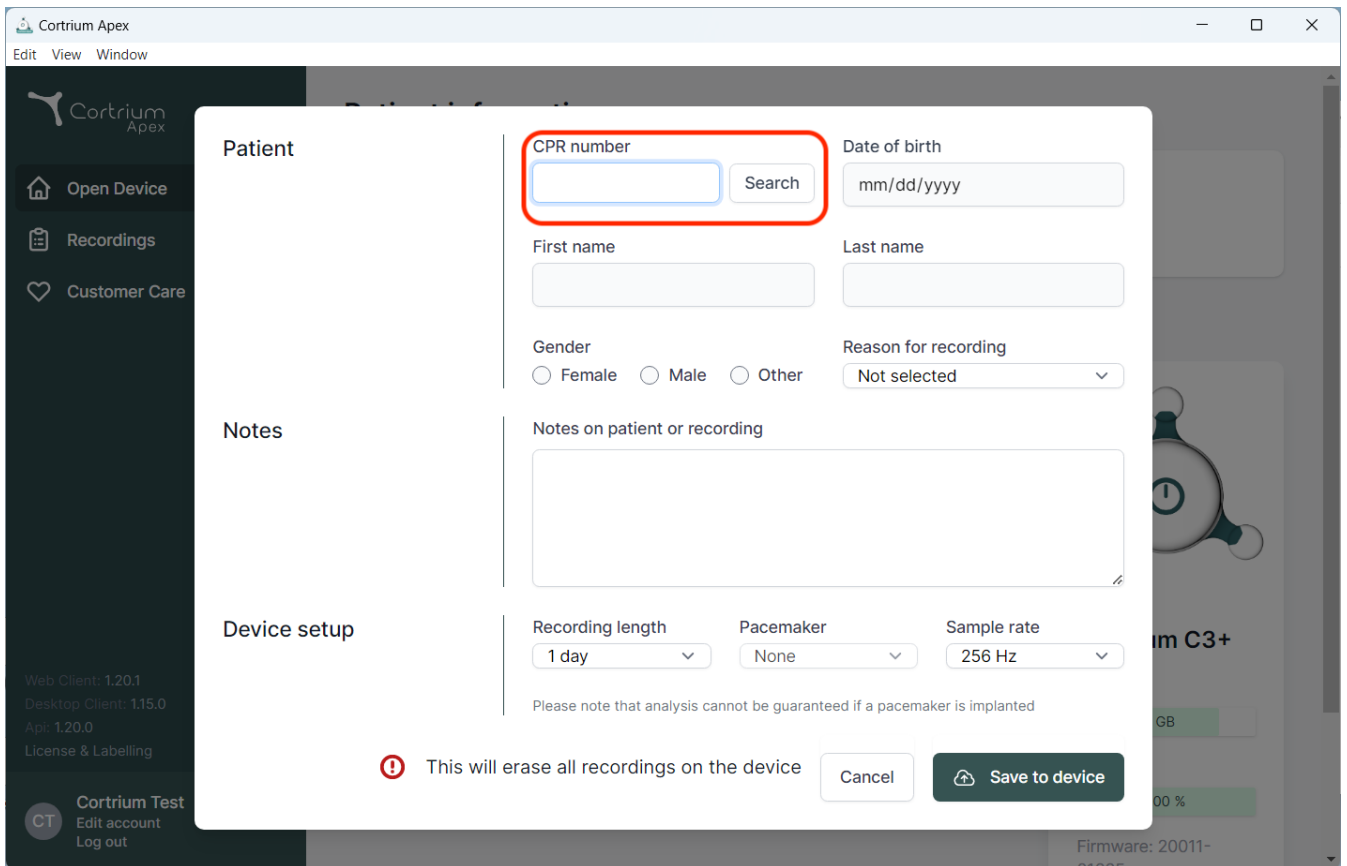
[Save changes](#)

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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Pacemaker

Sample rate

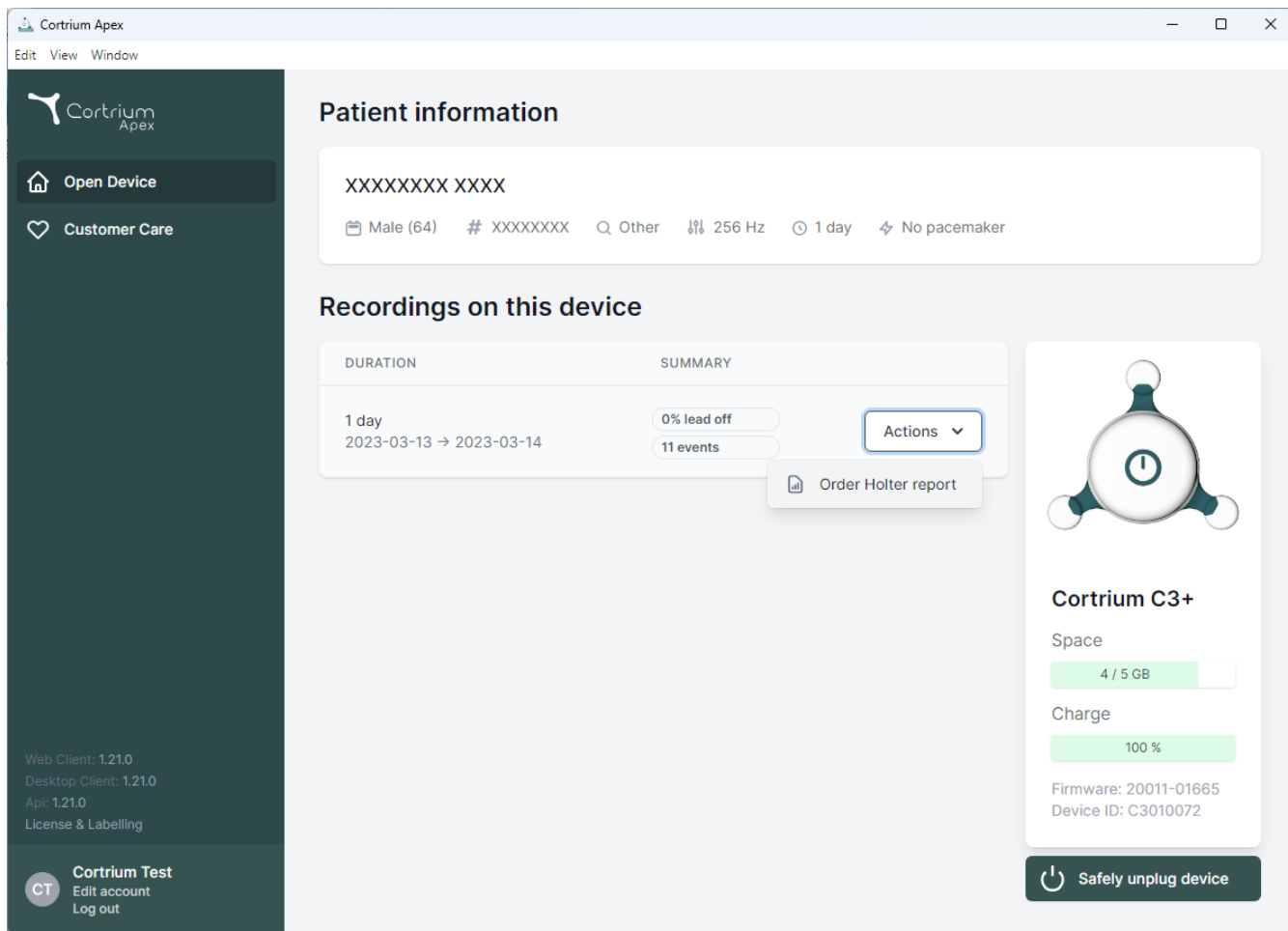
Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

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The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device" and "Customer Care". The main content area is divided into two sections: "Patient information" and "Recordings on this device".

Patient information

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions

[Order Holter report](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

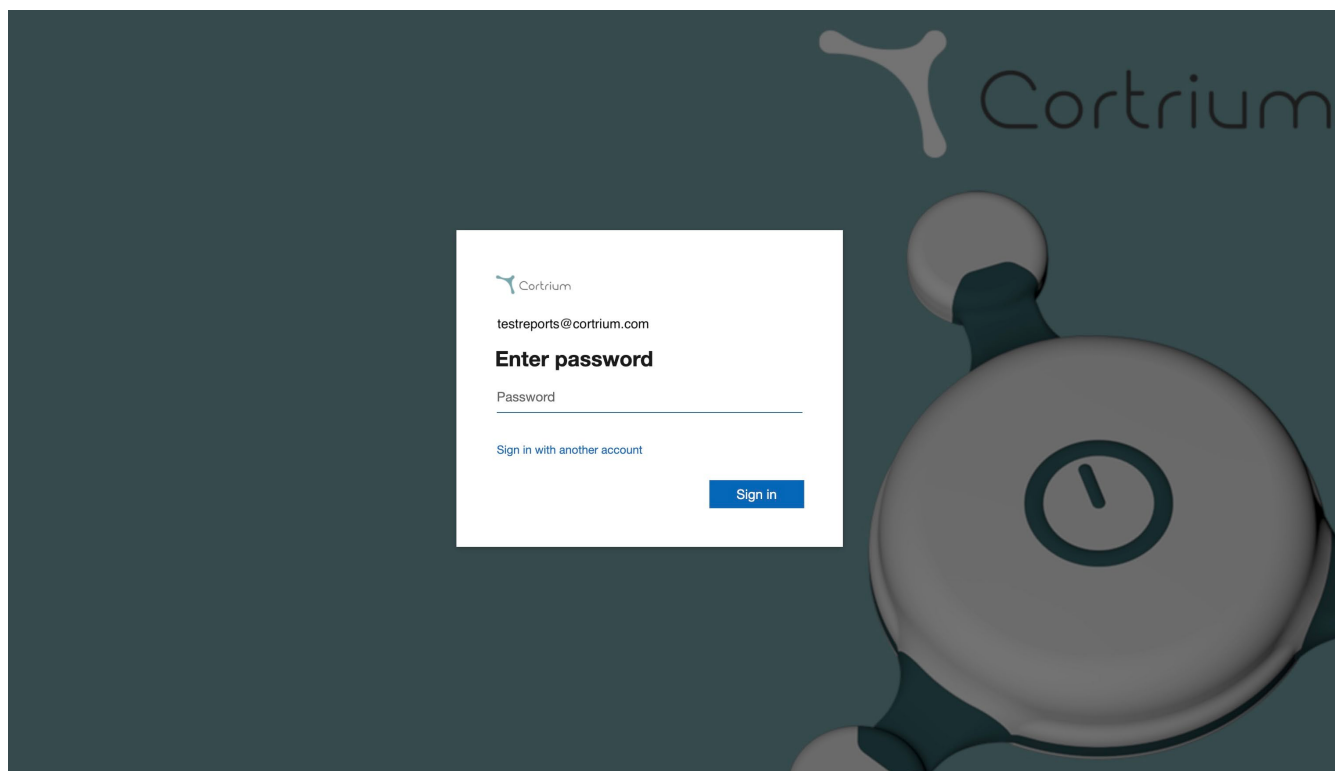
[Safely unplug device](#)

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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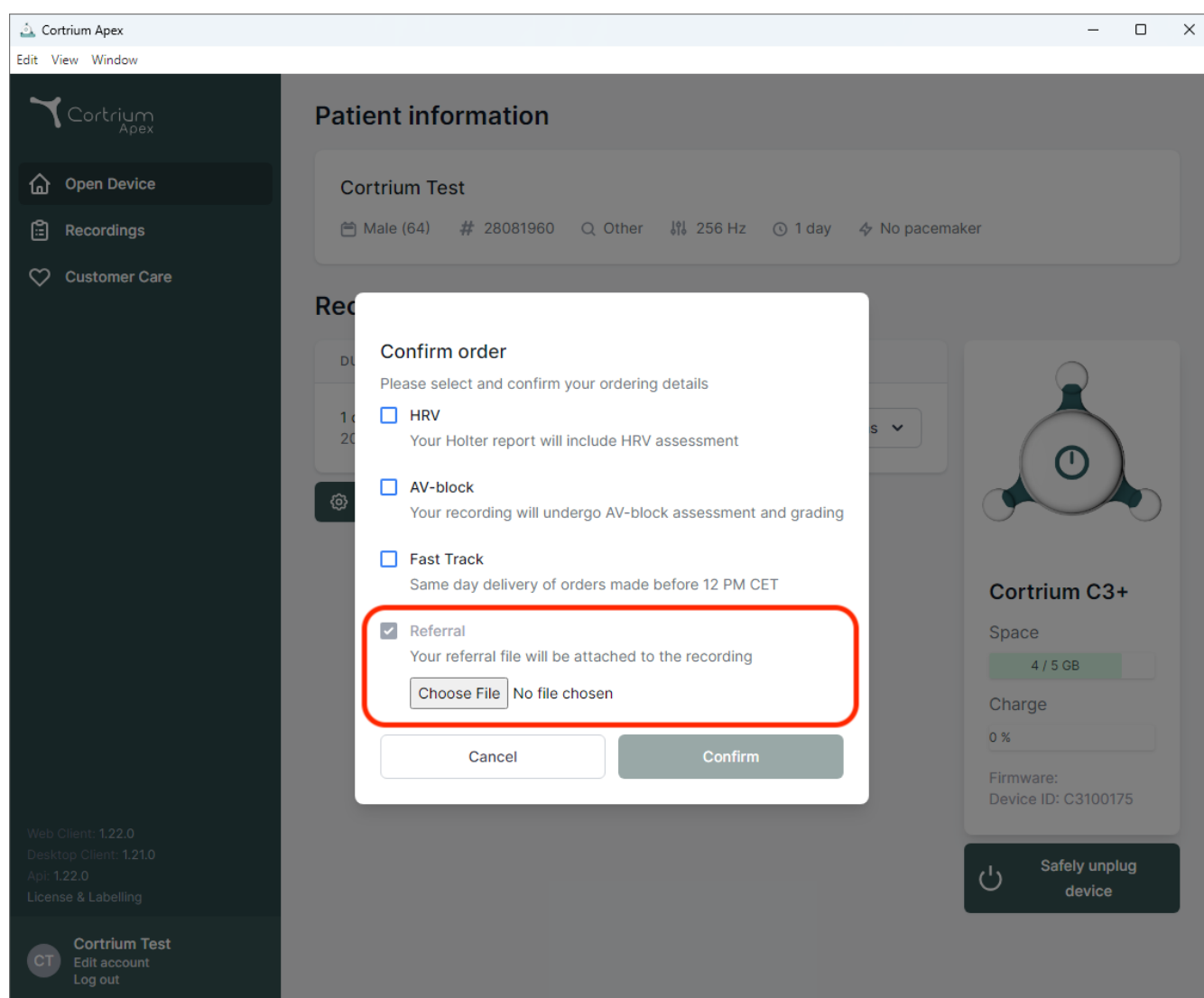
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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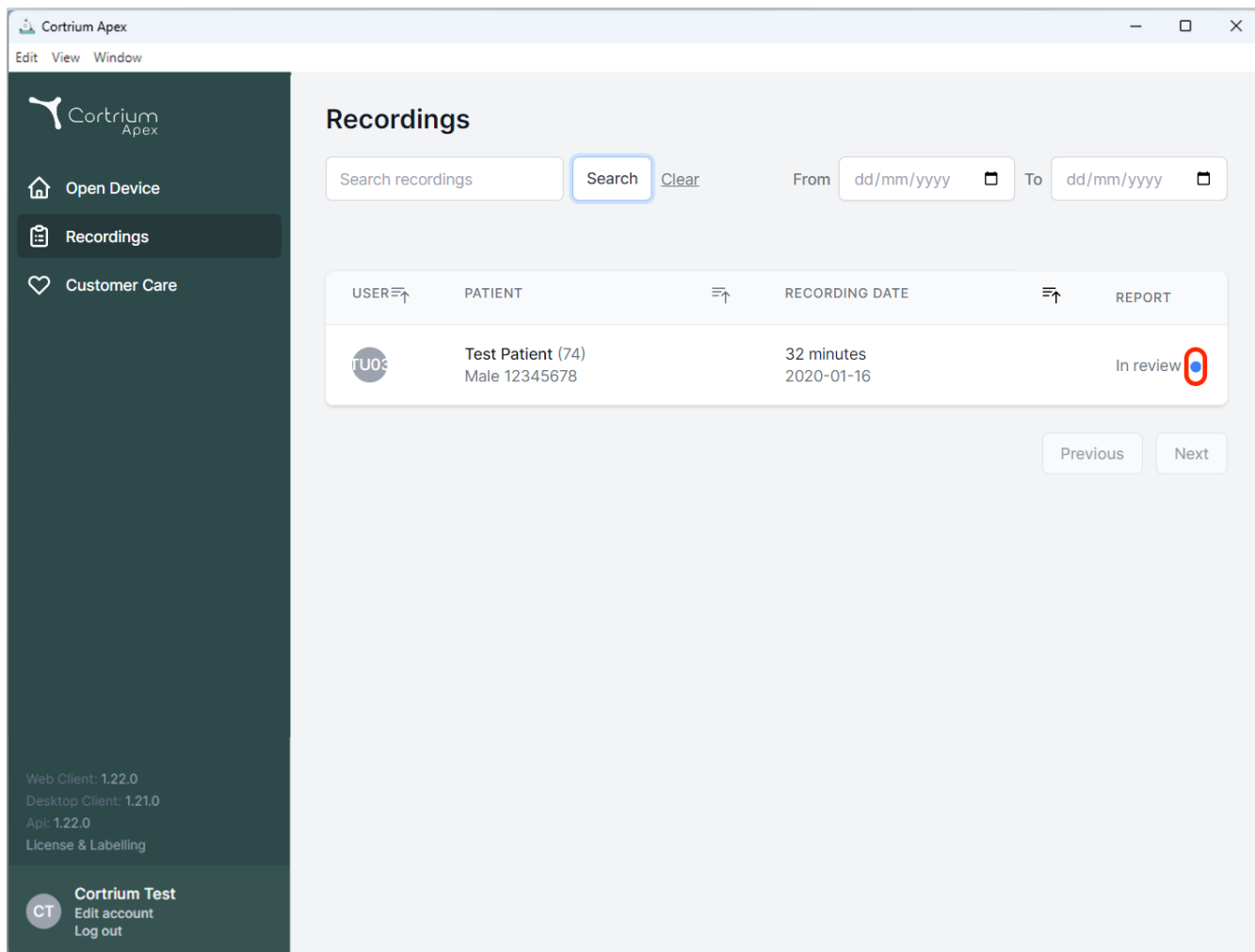
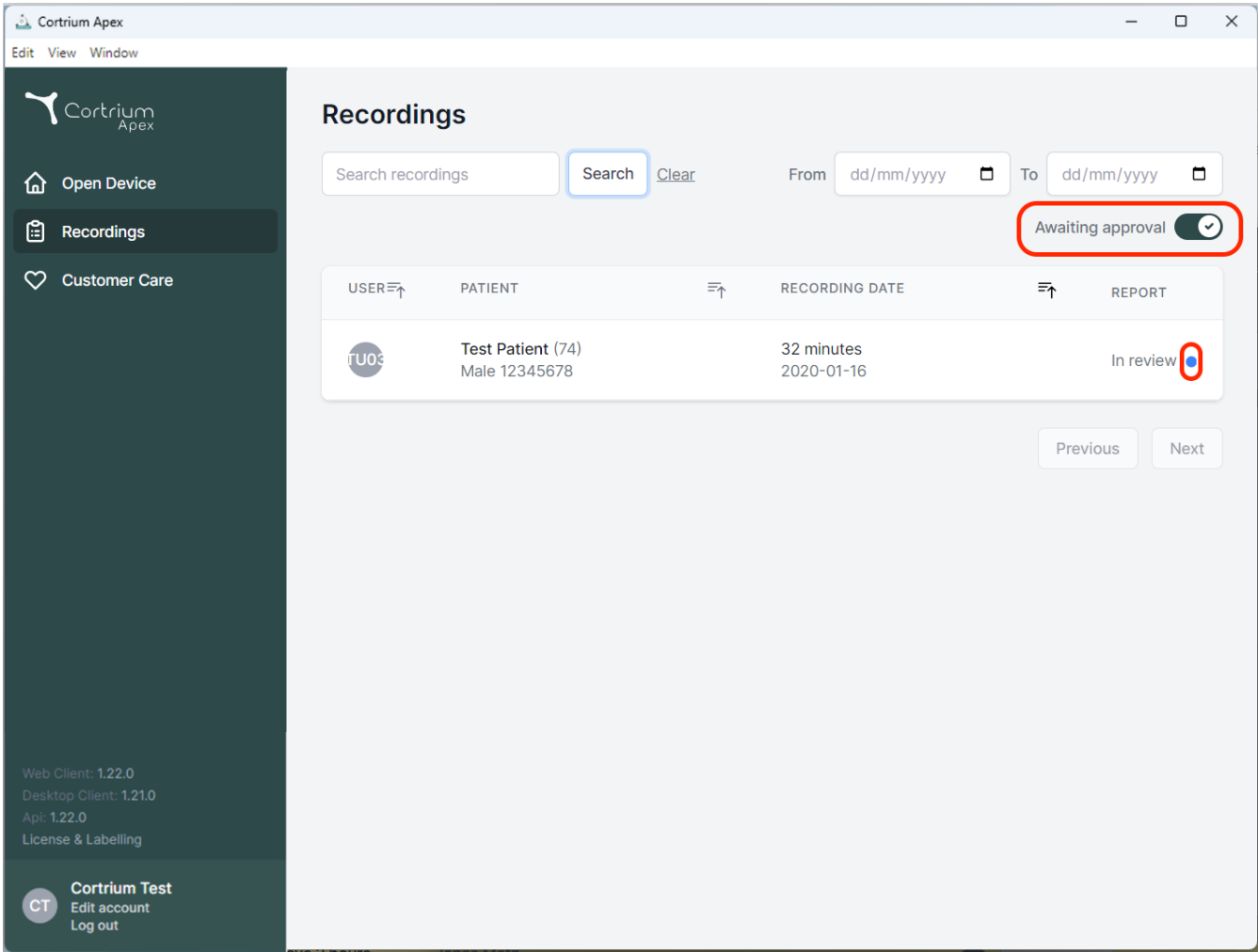
Edit View Window

Recordings

Search recordings Search Clear From dd/mm/yyyy To dd/mm/yyyy

Awaiting approval ☒


USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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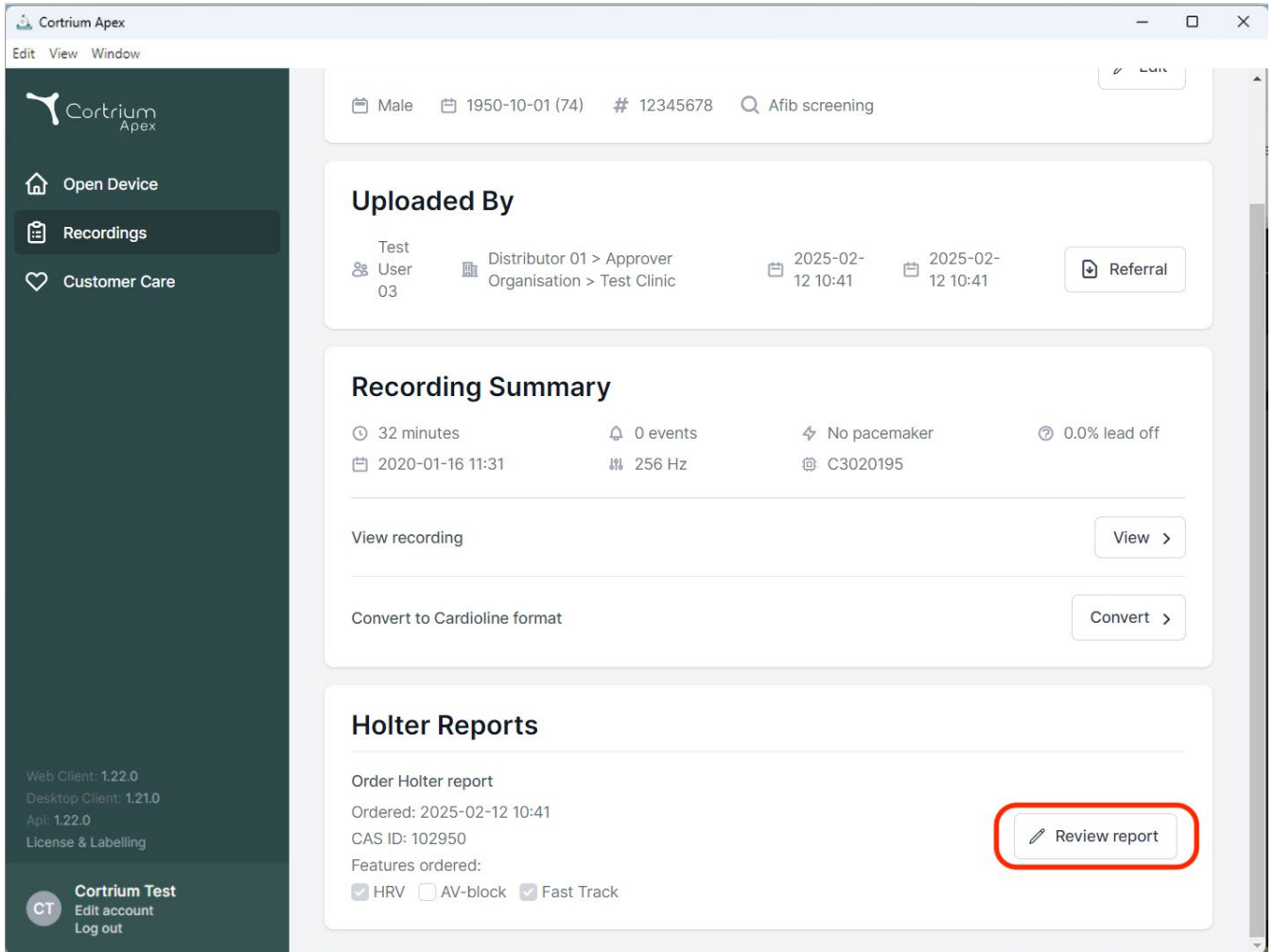
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes "Edit", "View", and "Window" options. The left sidebar contains the Cortrium Apex logo and three main menu items: "Open Device", "Recordings" (highlighted), and "Customer Care". Below these, version information is listed: "Web Client: 1.22.0", "Desktop Client: 1.21.0", "Apl: 1.22.0", and "License & Labelling". At the bottom of the sidebar, a "Cortrium Test" section includes "Edit account" and "Log out" links.

The main content area shows patient details at the top: "Male", "1950-10-01 (74)", "# 12345678", and "Afib screening". Below this, the "Uploaded By" section lists "Test User 03" as the distributor, with dates "2025-02-12 10:41" and a "Referral" button.

The "Recording Summary" section provides key metrics: "32 minutes", "0 events", "No pacemaker", and "0.0% lead off". It also shows the recording date "2020-01-16 11:31", a frequency of "256 Hz", and a device ID "C3020195". Action buttons for "View recording" and "Convert to Cardioline format" are present.

The "Holter Reports" section shows an "Order Holter report" with an order date of "2025-02-12 10:41" and a CAS ID of "102950". It lists ordered features: "HRV" (checked), "AV-block" (unchecked), and "Fast Track" (checked). A red box highlights the "Review report" button in the top right corner of this section.

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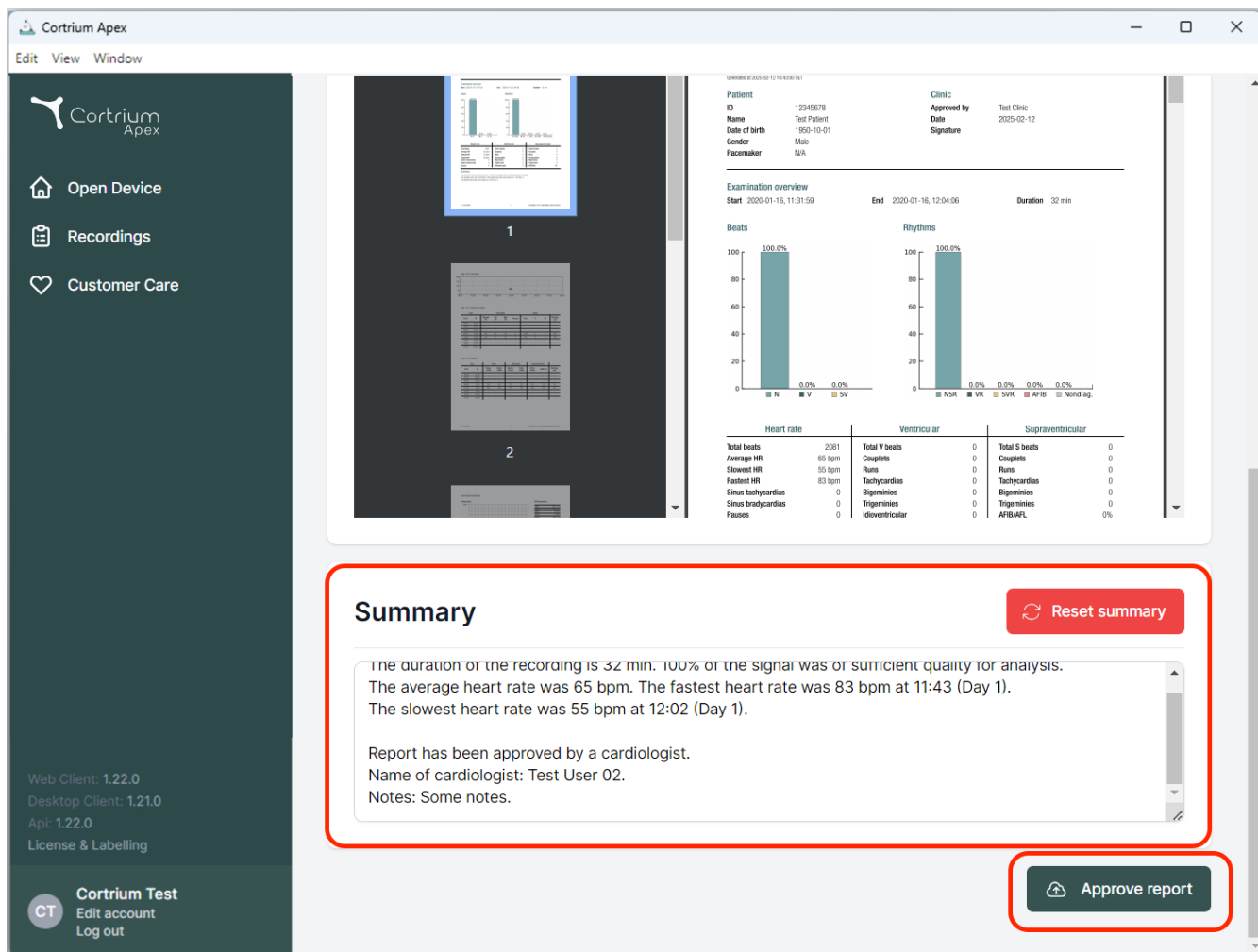The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area shows a patient report for 'Test Patient' (ID: 12345678, Date of birth: 1950-10-01, Gender: Male, Pacemaker: N/A). The report includes an 'Examination overview' section with start/end times and duration. Below this are two bar charts: 'Beats' (showing 100.0% for N, 0.0% for V, 0.0% for SV) and 'Rhythms' (showing 100.0% for NSR, 0.0% for VR, 0.0% for SVR, 0.0% for AFB, 0.0% for Nonrhythm). A table at the bottom provides detailed heart rate and rhythm statistics. A red box highlights the 'Summary' section, which contains a 'Reset summary' button and a text area with the following text: 'The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis. The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1). The slowest heart rate was 55 bpm at 12:02 (Day 1). Report has been approved by a cardiologist. Name of cardiologist: Test User 02. Notes: Some notes.' Another red box highlights the 'Approve report' button at the bottom right.

Summary Reset summary

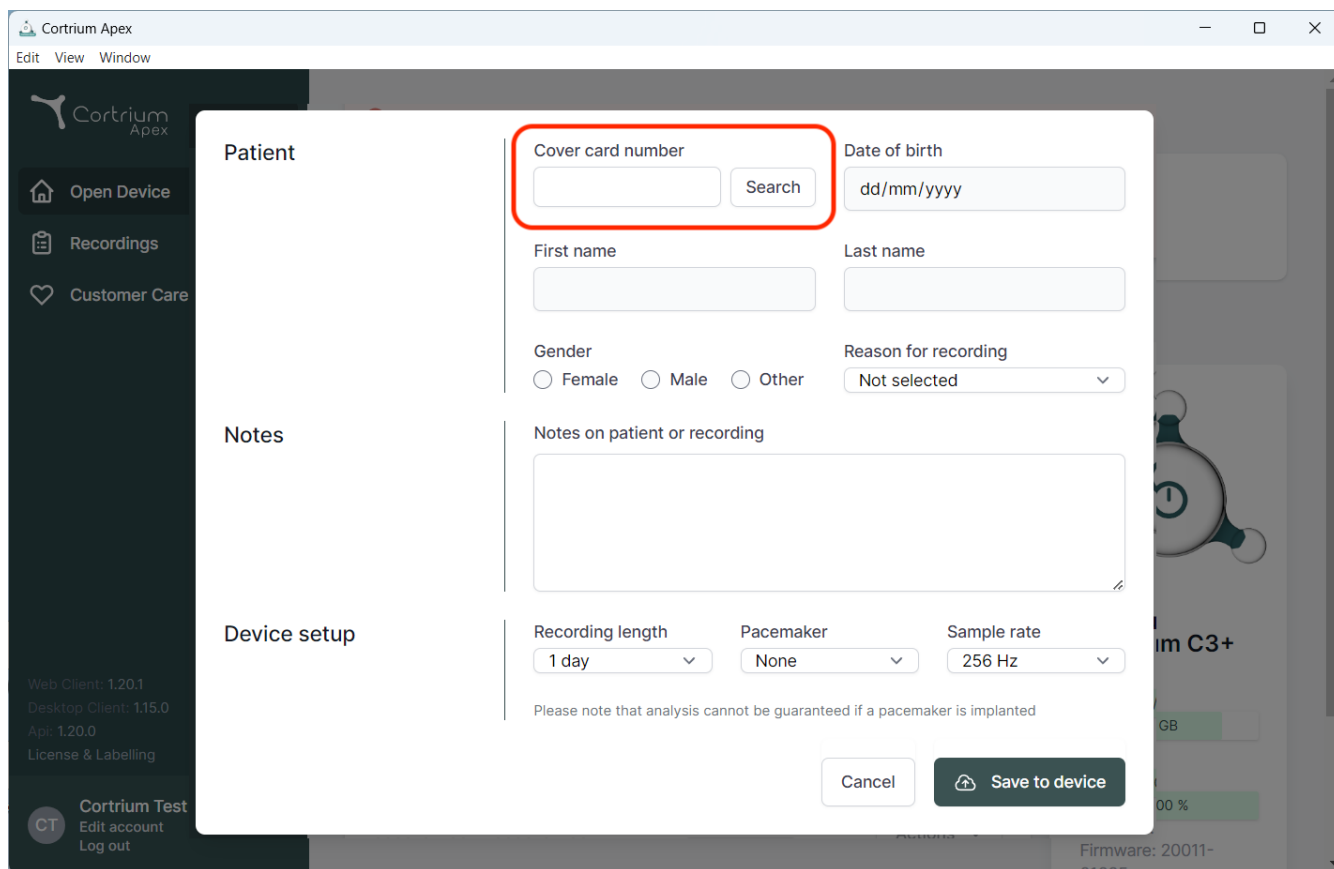
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
 Name of cardiologist: Test User 02.
 Notes: Some notes.

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Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Cancel Save to device

02/2025

Cortrium ApS - Erik Husfeldts Vej 7 - DK-2630 Taastrup - Danemarca
E-mail: info@cortrium.com - Website : www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123
46 / 46