Cortrium Apex 1.27

Cortrium Apex 1.27

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
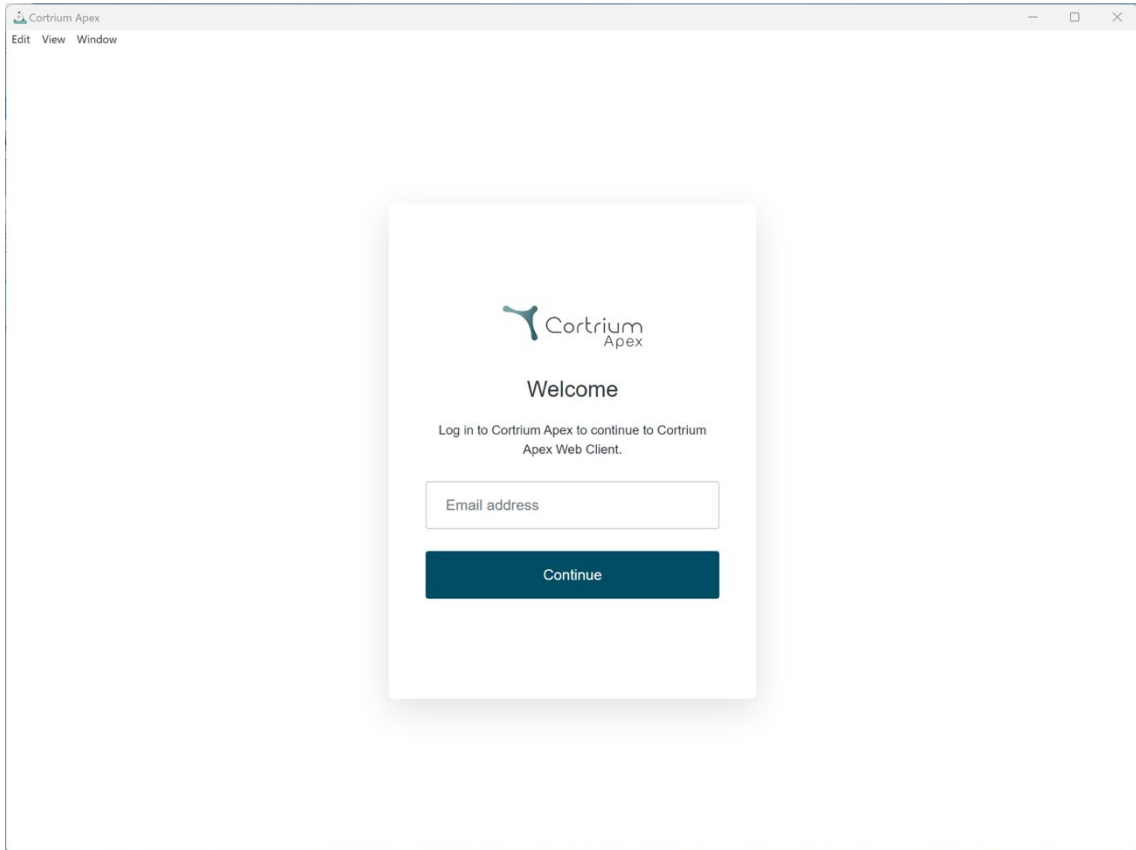
The screenshot shows a web browser window titled "Cortrium Apex" with a menu bar containing "Edit", "View", and "Window". In the center of the browser is a white login dialog box. The dialog box features the Cortrium Apex logo at the top, followed by the heading "Welcome". Below the heading is the instruction "Log in to Cortrium Apex to continue to Cortrium Apex Web Client." There is a text input field labeled "Email address" and a dark blue button labeled "Continue" below it.

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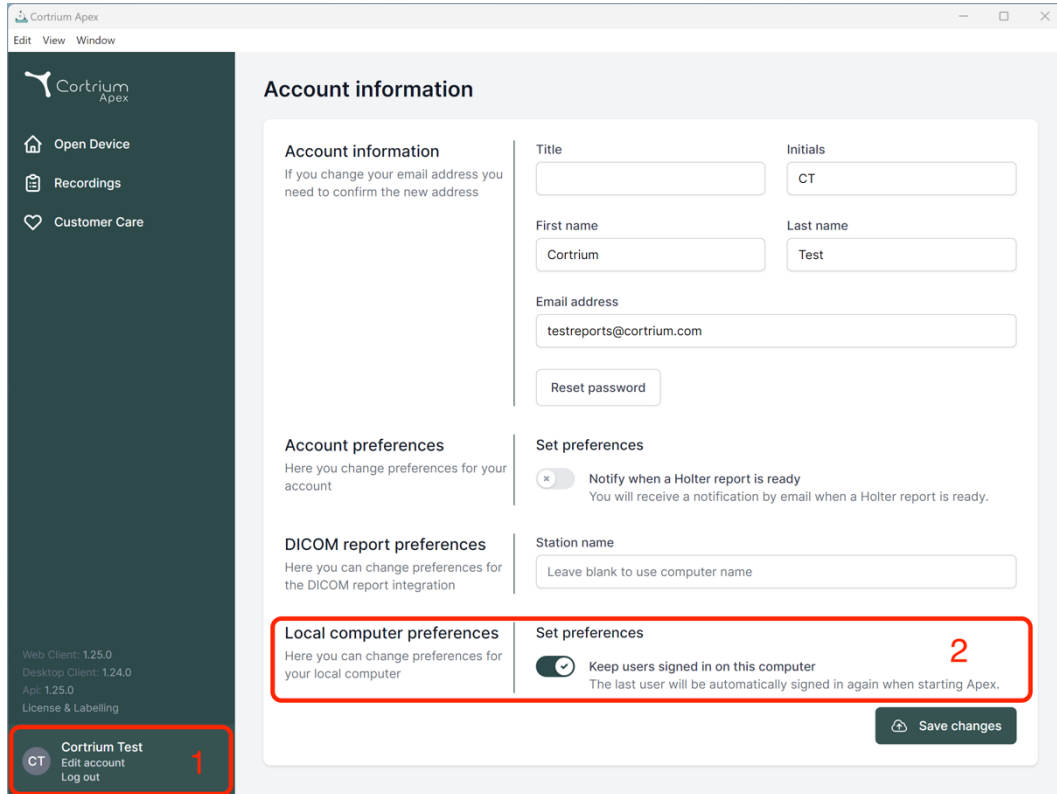
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client, Desktop Client, and API, along with a 'License & Labelling' link. At the bottom of the sidebar is a user profile section for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The main content area is titled 'Account information' and contains several sections: 'Account information' (with fields for Title, Initials, First name, Last name, and Email address), 'Account preferences' (with a 'Reset password' button), 'DICOM report preferences', 'Local computer preferences' (highlighted with a red box and a red '1'), and 'Set preferences' (highlighted with a red box and a red '2'). The 'Set preferences' section includes a toggle for 'Notify when a Holter report is ready' and a 'Station name' field. A 'Save changes' button is located at the bottom right of the main content area.

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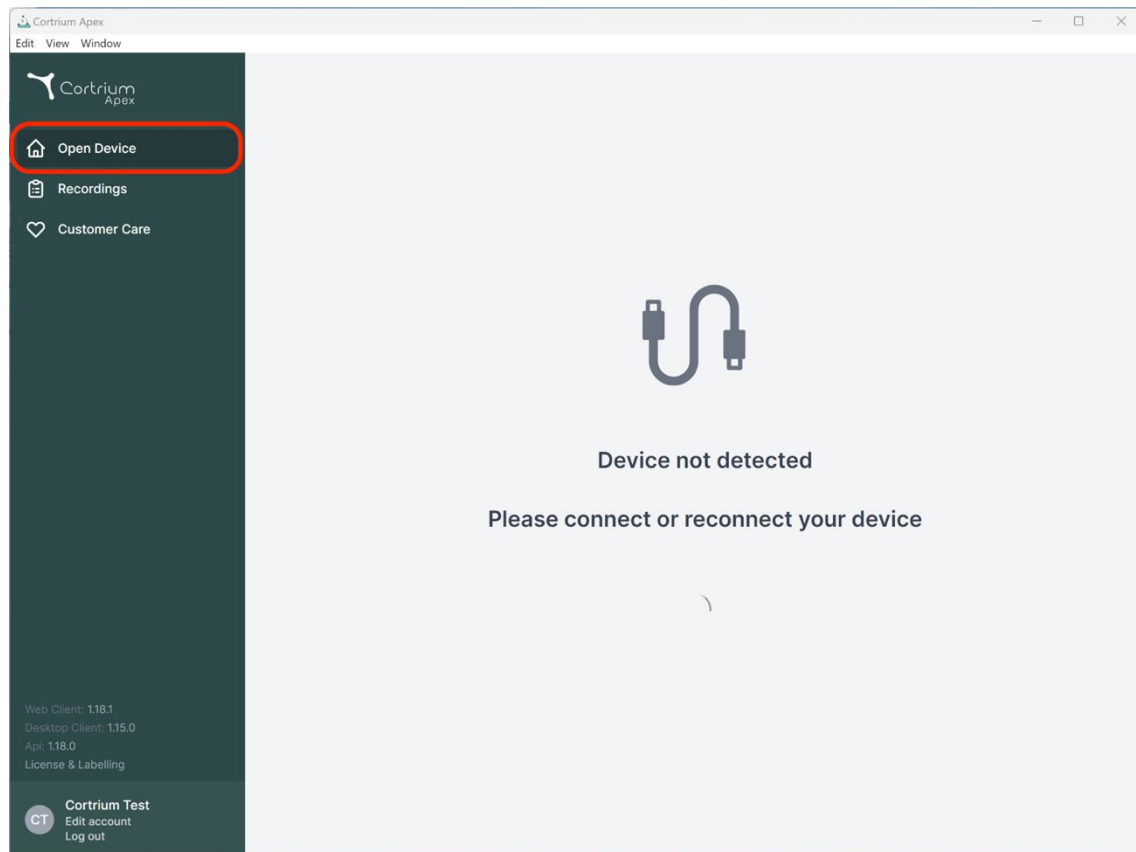
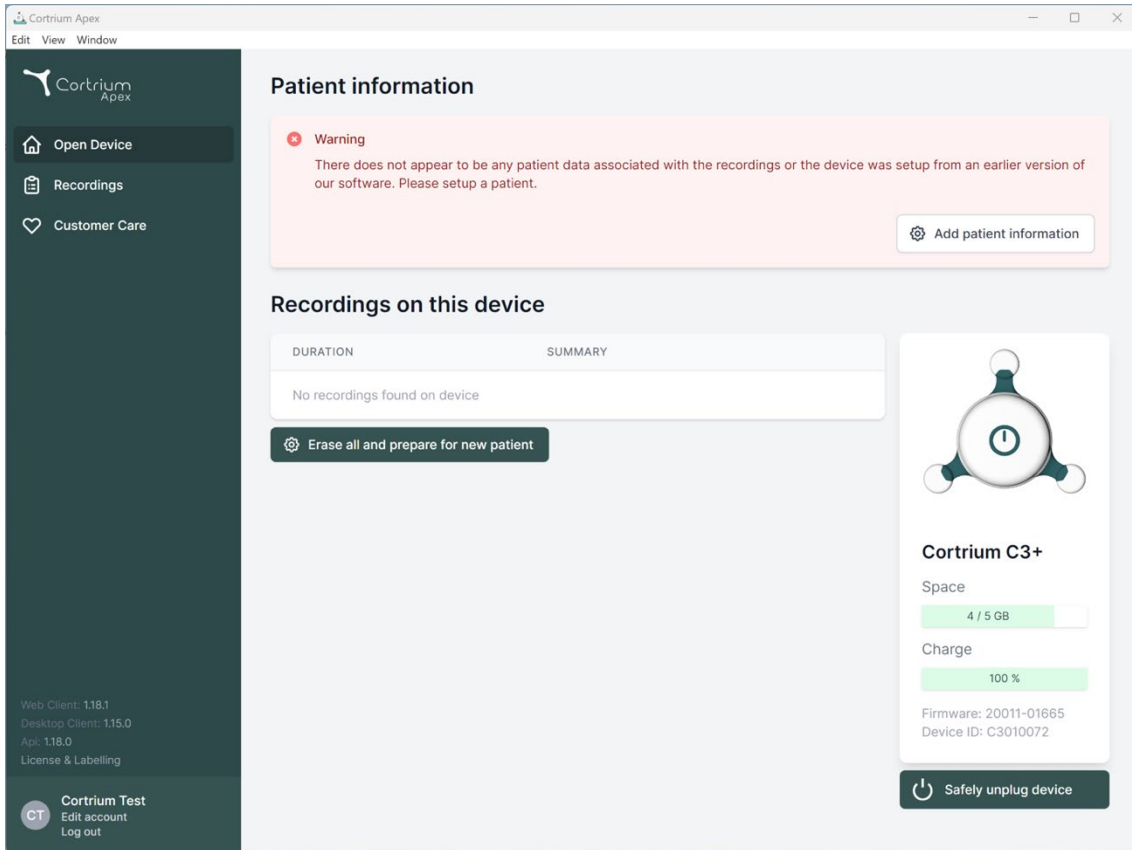
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a user profile section for 'Cortrium Test' with 'Edit account' and 'Log out' options. The main content area is titled 'Patient information' and features a red warning box stating: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table showing 'DURATION' and 'SUMMARY' columns, currently displaying 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' shows 'Space' at 4 / 5 GB and 'Charge' at 100 %, along with firmware and device ID information, and a 'Safely unplug device' button.

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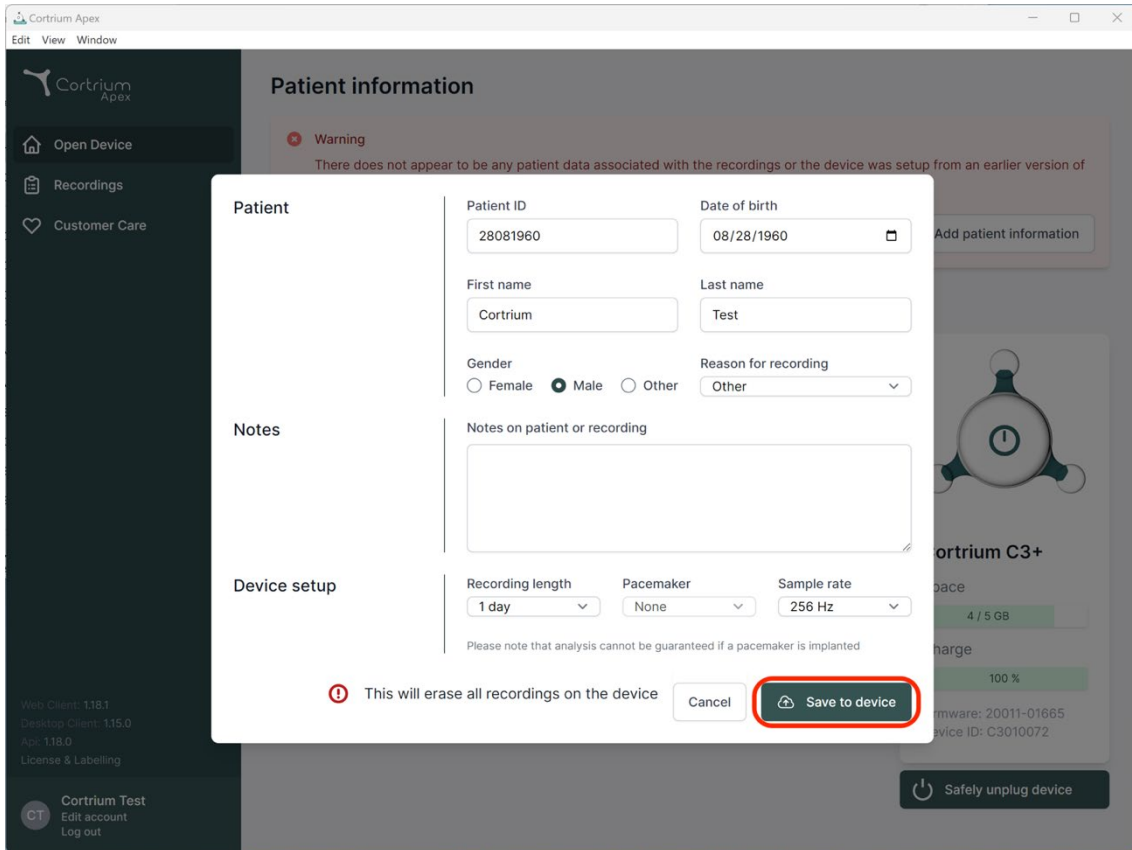Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

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
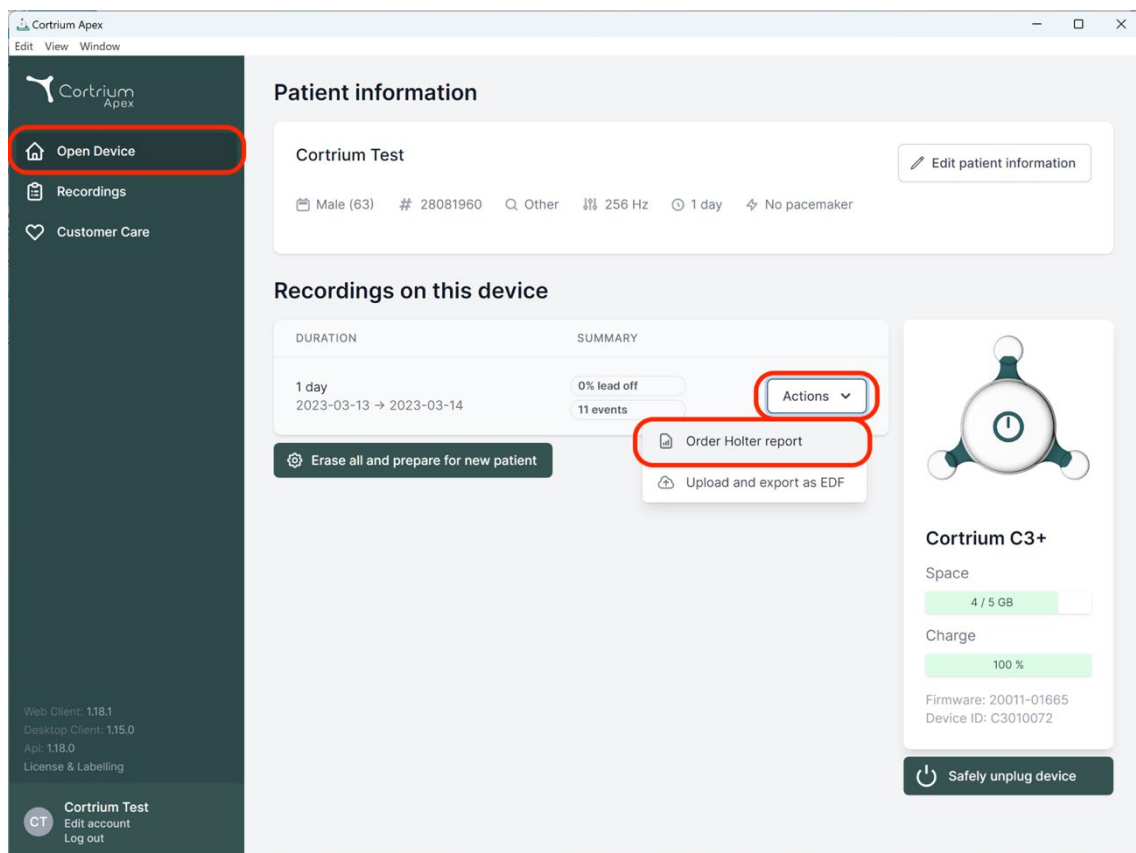
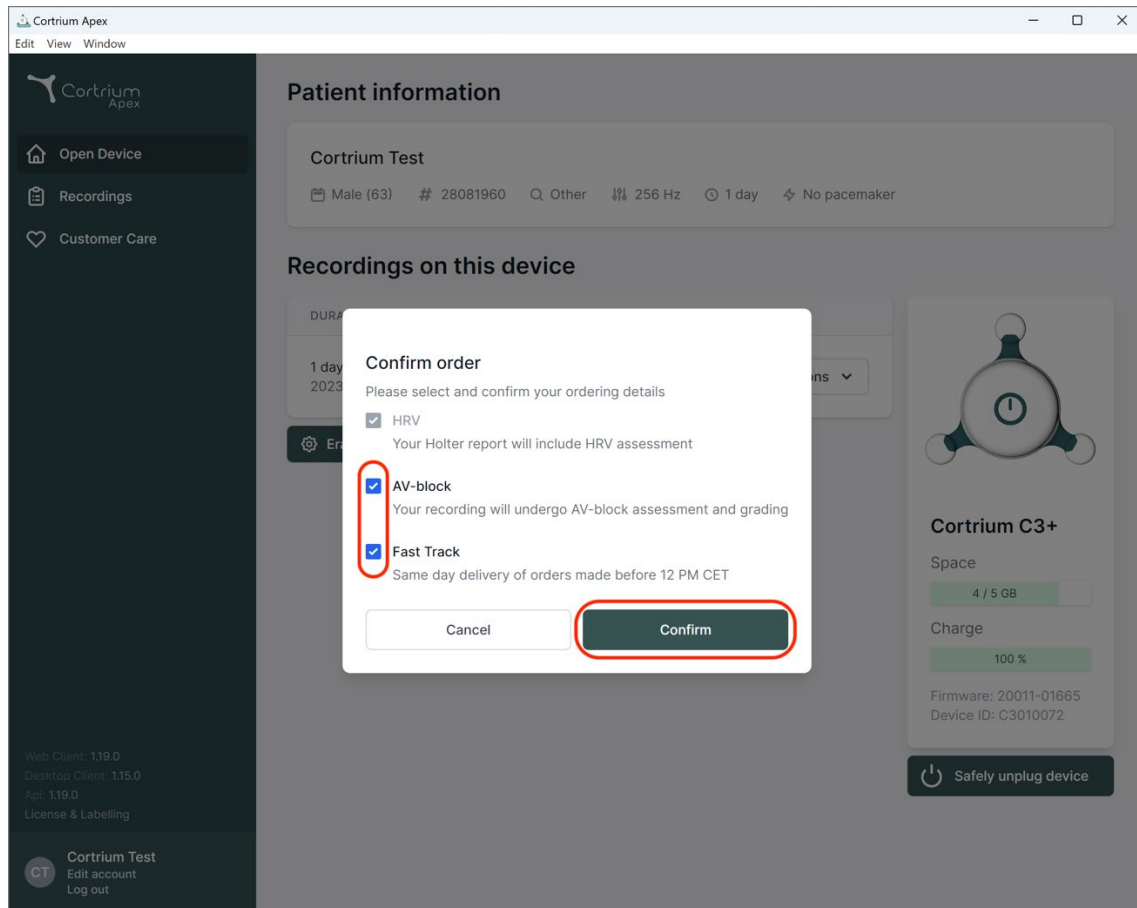
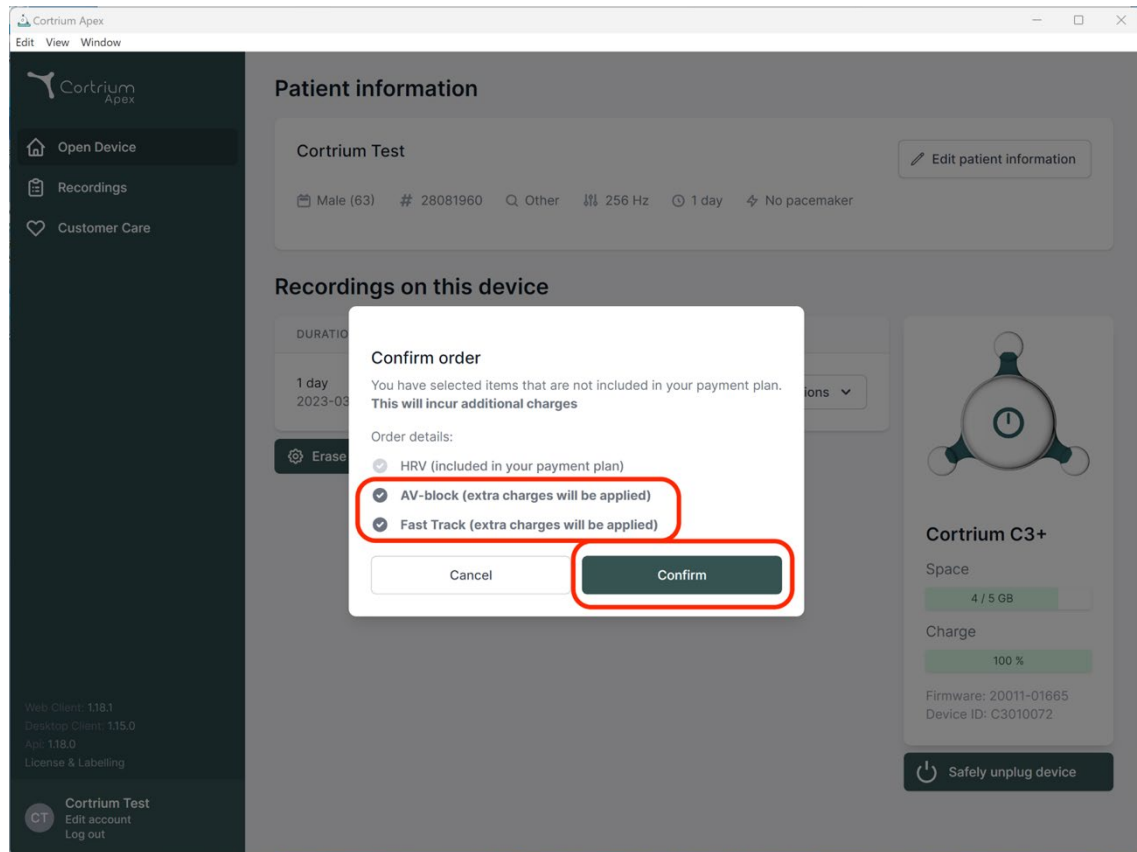
Inspelningen laddas nu upp.

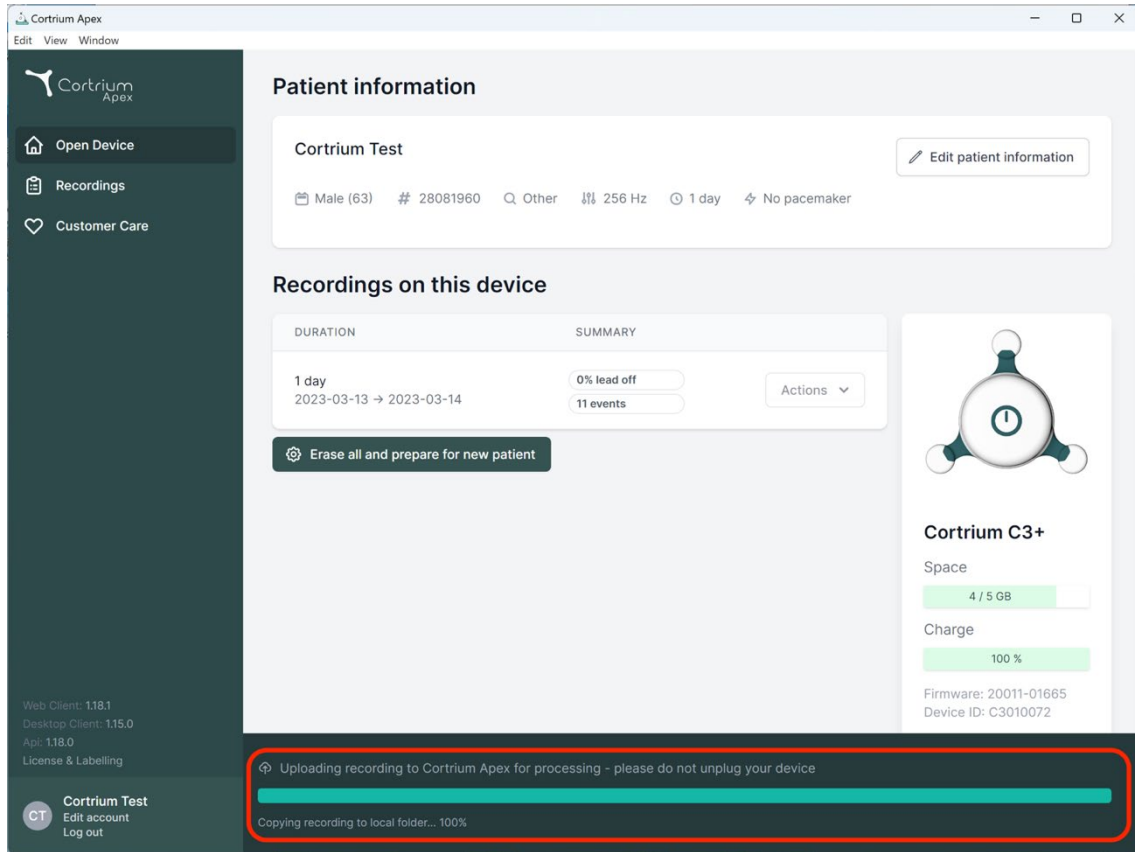The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and recording duration (1 day). Below this is a 'Recordings on this device' section with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A red box highlights a status bar at the bottom of the interface, which contains the text: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. To the right of the recording table, there is a device status card for 'Cortrium C3+' showing 4/5 GB space and 100% charge.

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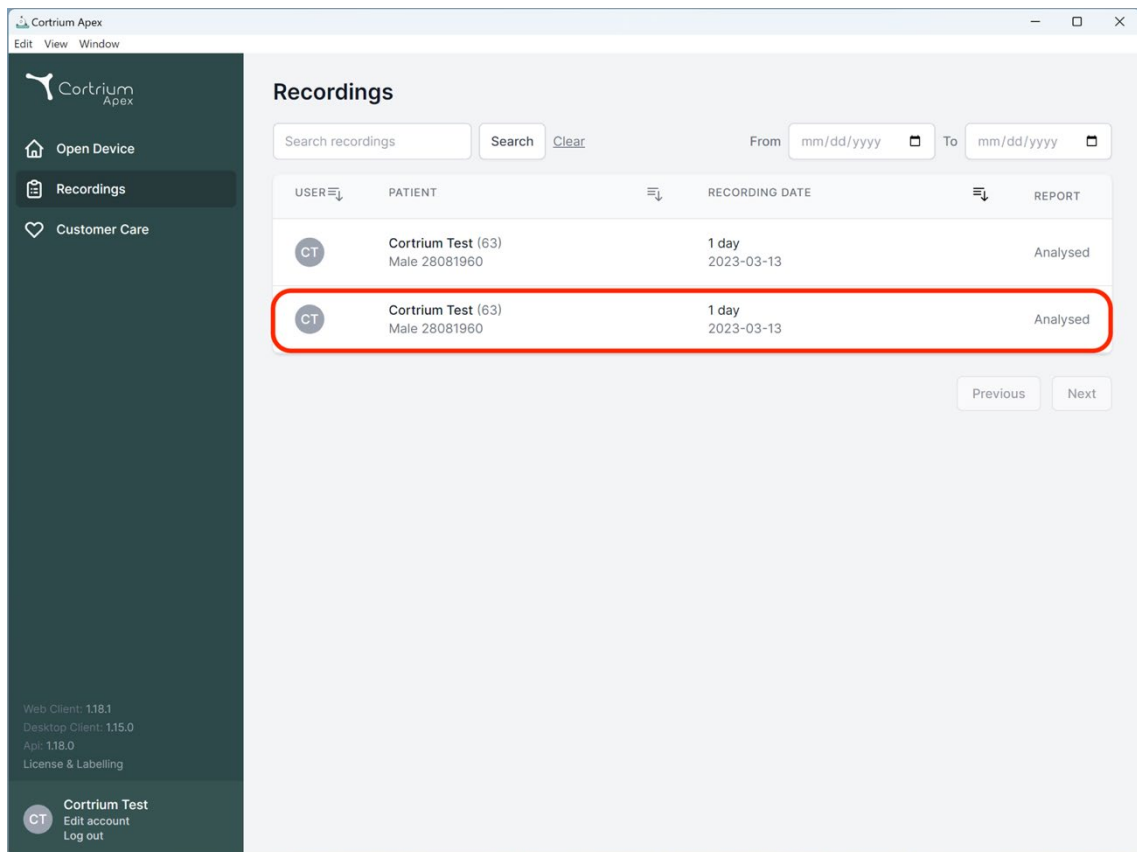
Recordings

Search recordings Search Clear From mm/dd/yyyy To mm/dd/yyyy

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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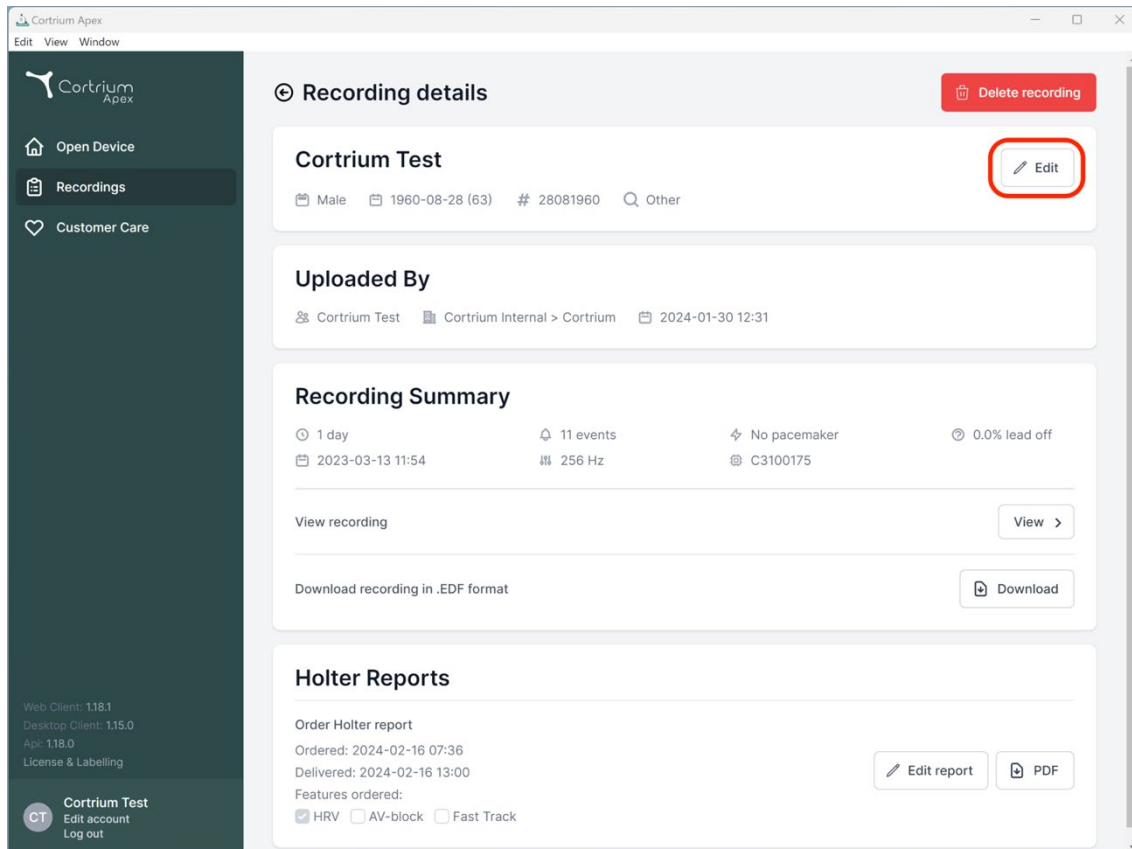
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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
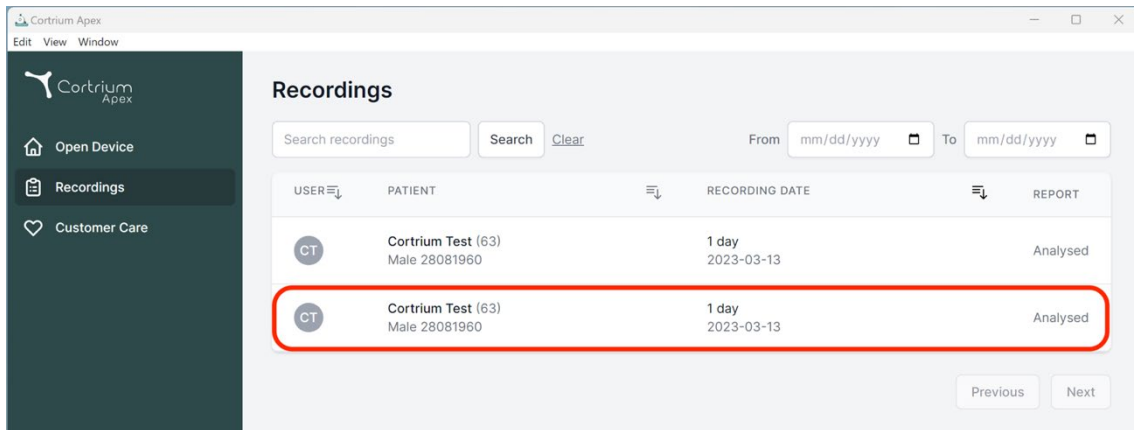
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. On the left is a sidebar with 'Open Device', 'Recordings', and 'Customer Care'. The main area has a search bar and a table of recordings. The table has columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row is highlighted with a red border.

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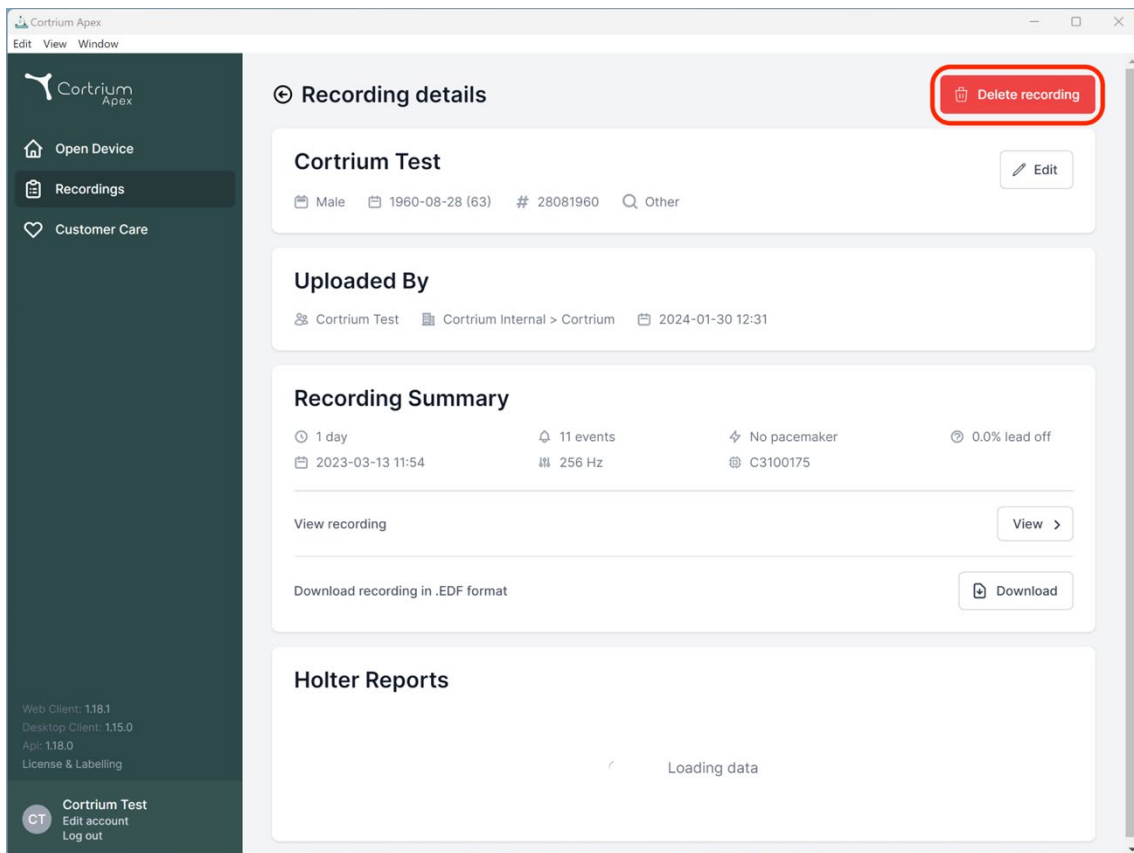The screenshot shows the 'Recording details' page for 'Cortrium Test'. The 'Delete recording' button is highlighted with a red rectangle. The page displays patient information, upload details, a recording summary, and a section for Holter reports.

Recording details

Cortrium Test

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View

Download recording in .EDF format Download

Holter Reports

Loading data

Cortrium Apex
Edit View Window

 Cortrium Apex

 Open Device

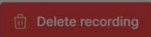
 Recordings

 Customer Care

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

 Cortrium Test
Edit account
Log out

Recording details

Cortrium Test 

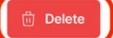
 Male  1960-08-28 (63)  # 28081960  Other


Uploaded By

 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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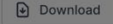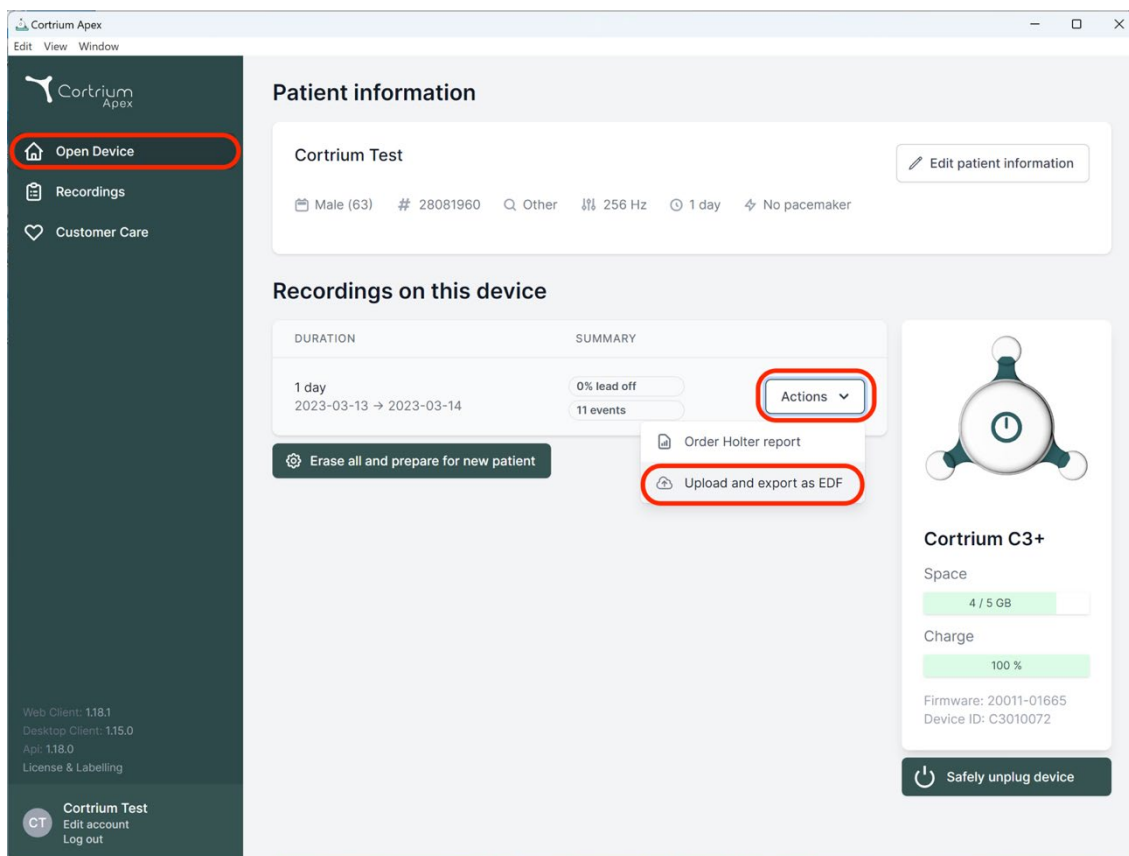
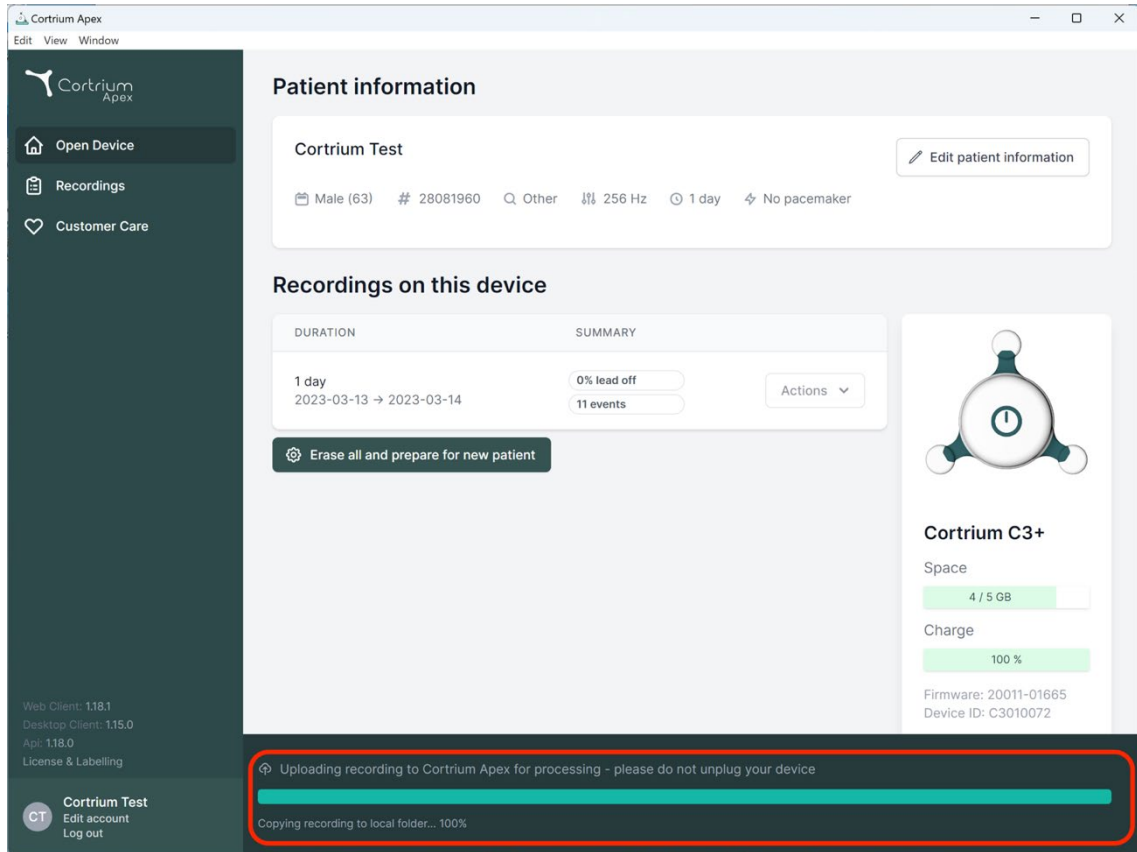
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

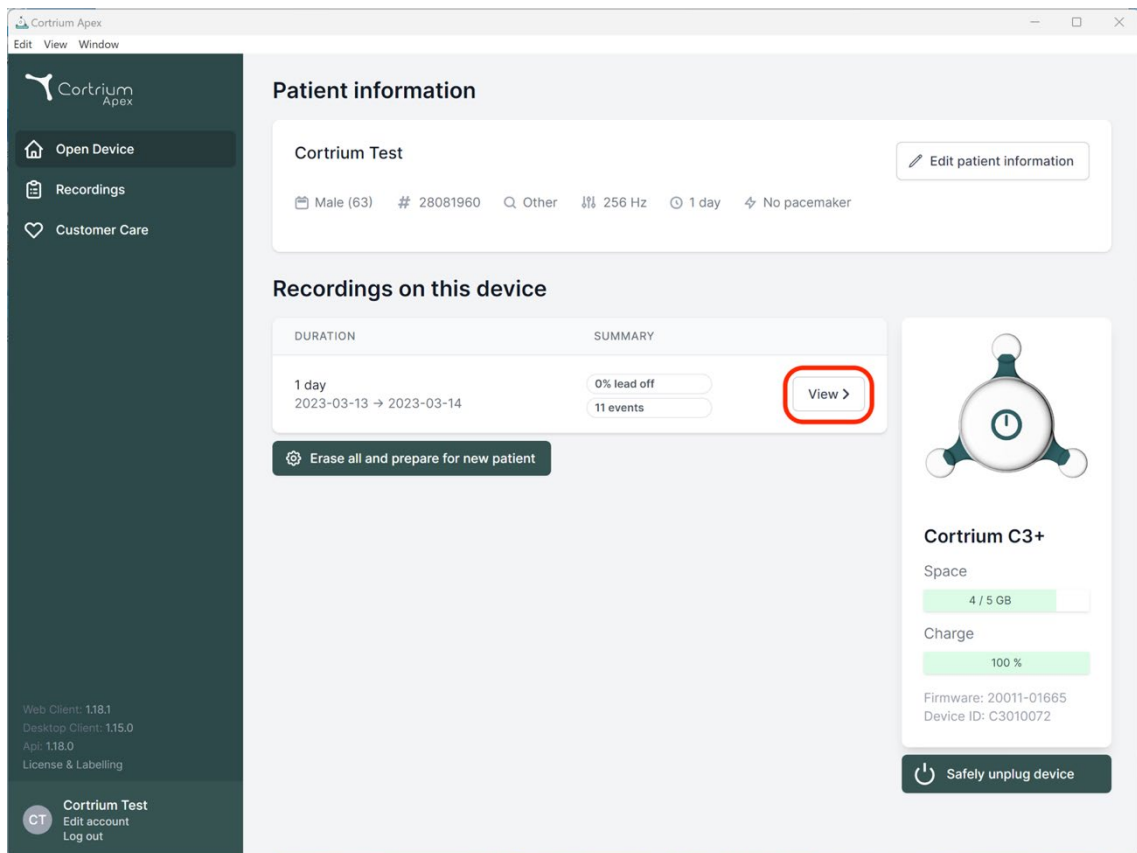
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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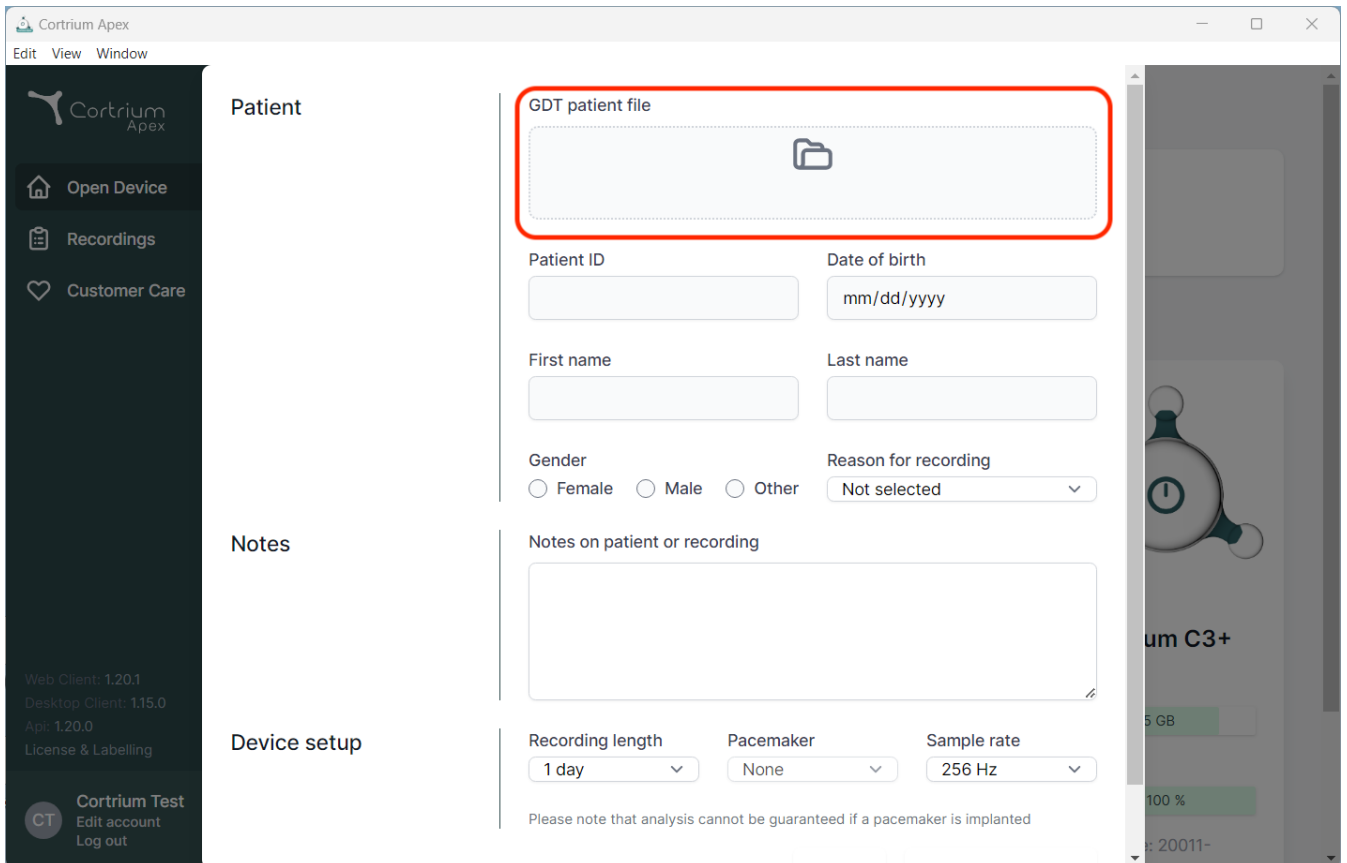
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains the Cortrium Apex logo, navigation links for 'Open Device', 'Recordings', and 'Customer Care', and version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0, License & Labelling). The main content area is divided into three sections: 'Patient', 'Notes', and 'Device setup'. The 'Patient' section contains a 'GDT patient file' upload field (highlighted with a red box), 'Patient ID' and 'Date of birth' input fields, 'First name' and 'Last name' input fields, 'Gender' radio buttons (Female, Male, Other), and a 'Reason for recording' dropdown menu. The 'Notes' section contains a 'Notes on patient or recording' text area. The 'Device setup' section contains 'Recording length', 'Pacemaker', and 'Sample rate' dropdown menus. A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'.

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Web Client: 1.20.1
Desktop Client: 1.15.0
Api: 1.20.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient

GDT patient file

Patient ID

Date of birth

mm/dd/yyyy

First name

Last name

Gender

☐ Female ☐ Male ☐ Other

Reason for recording

Not selected

Notes

Notes on patient or recording

Device setup

Recording length

1 day

Pacemaker

None

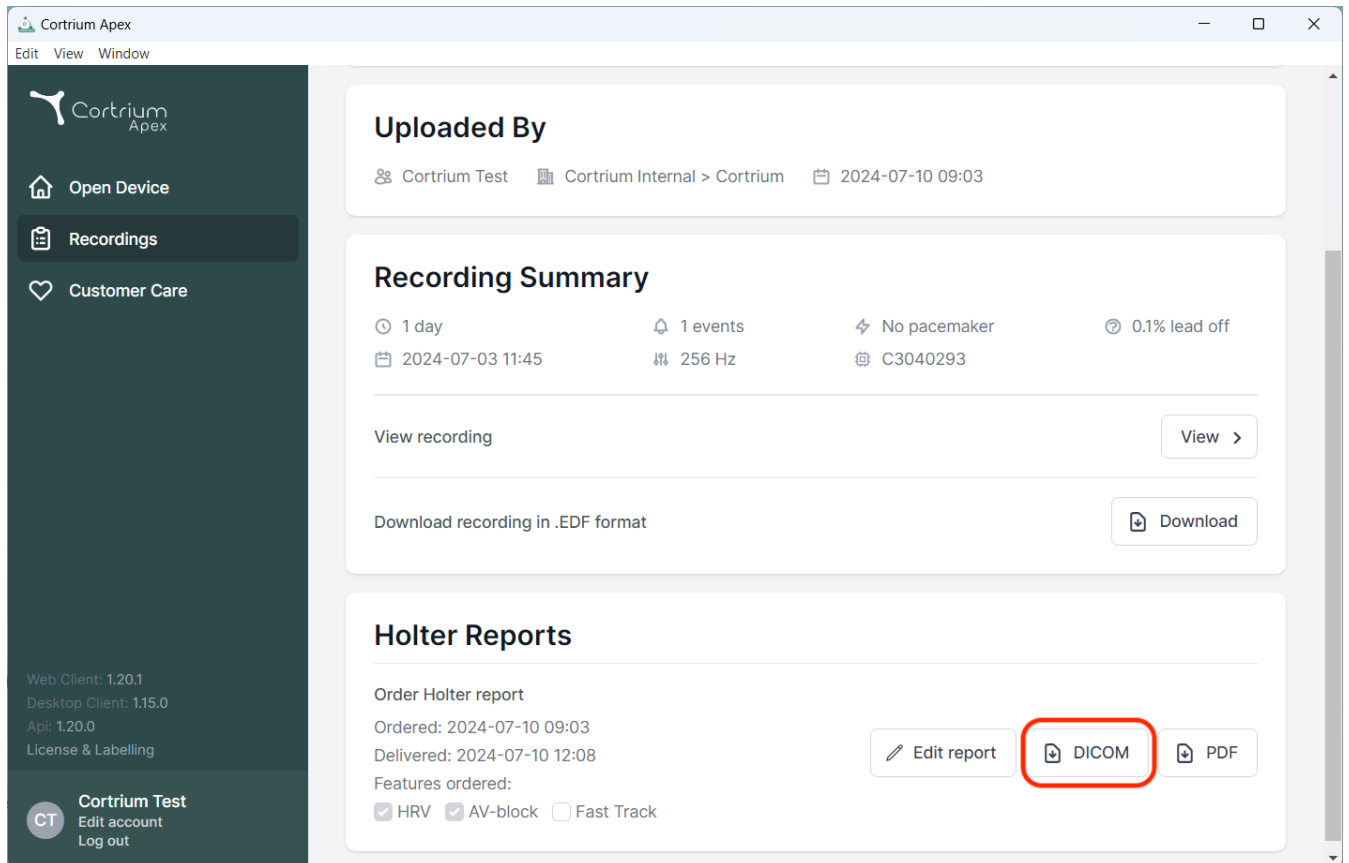
Sample rate

256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (labeled 'Download recording in .EDF format').
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

At the bottom left of the sidebar, version information is listed: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'Api: 1.20.0', and 'License & Labelling'. A user profile for 'Cortrium Test' is also visible.

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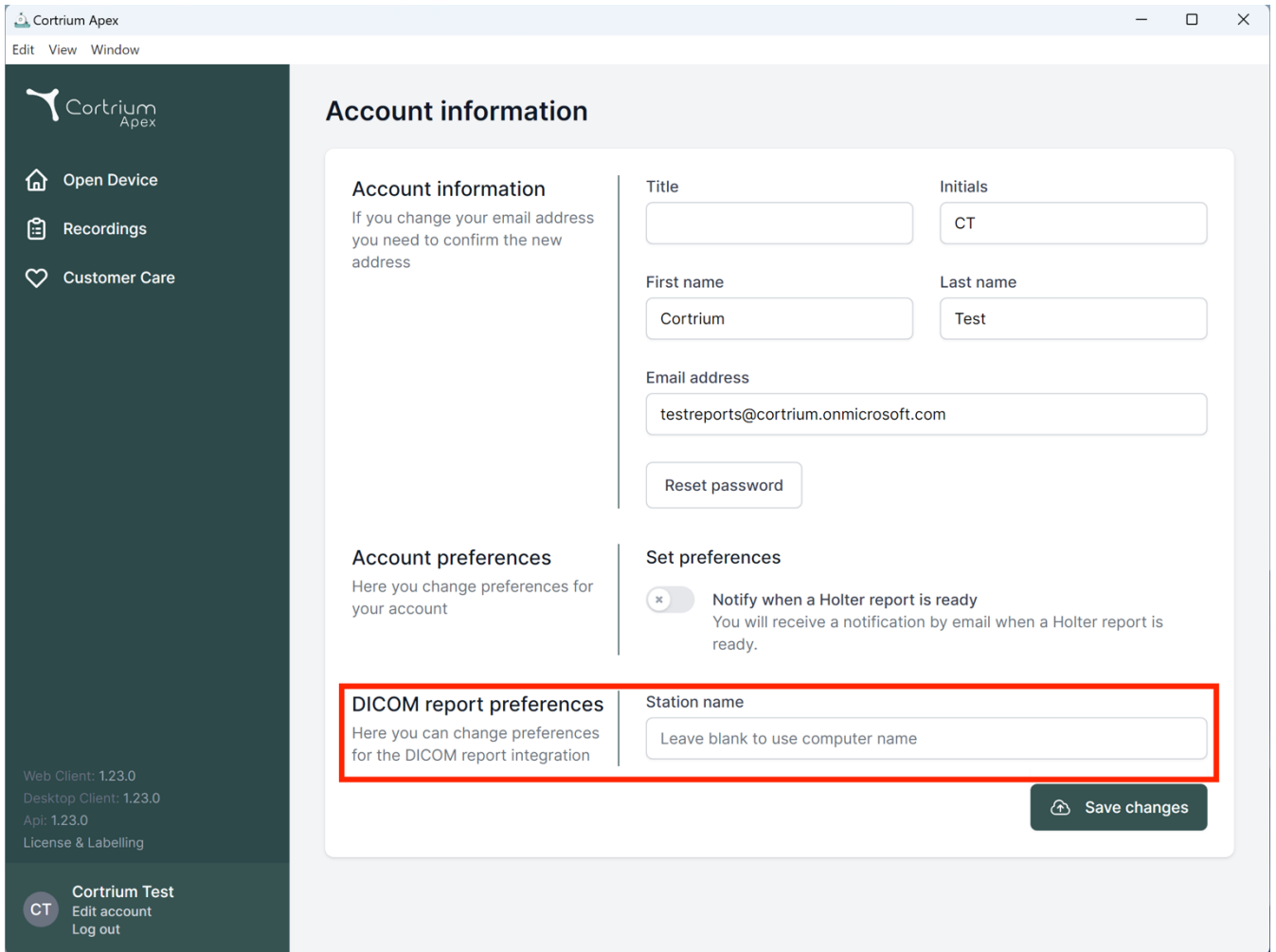Cortrium Apex
Edit View Window

Account information
If you change your email address you need to confirm the new address

Account preferences
Here you change preferences for your account

Set preferences
☐ Notify when a Holter report is ready
You will receive a notification by email when a Holter report is ready.

DICOM report preferences
Here you can change preferences for the DICOM report integration

Station name
Leave blank to use computer name

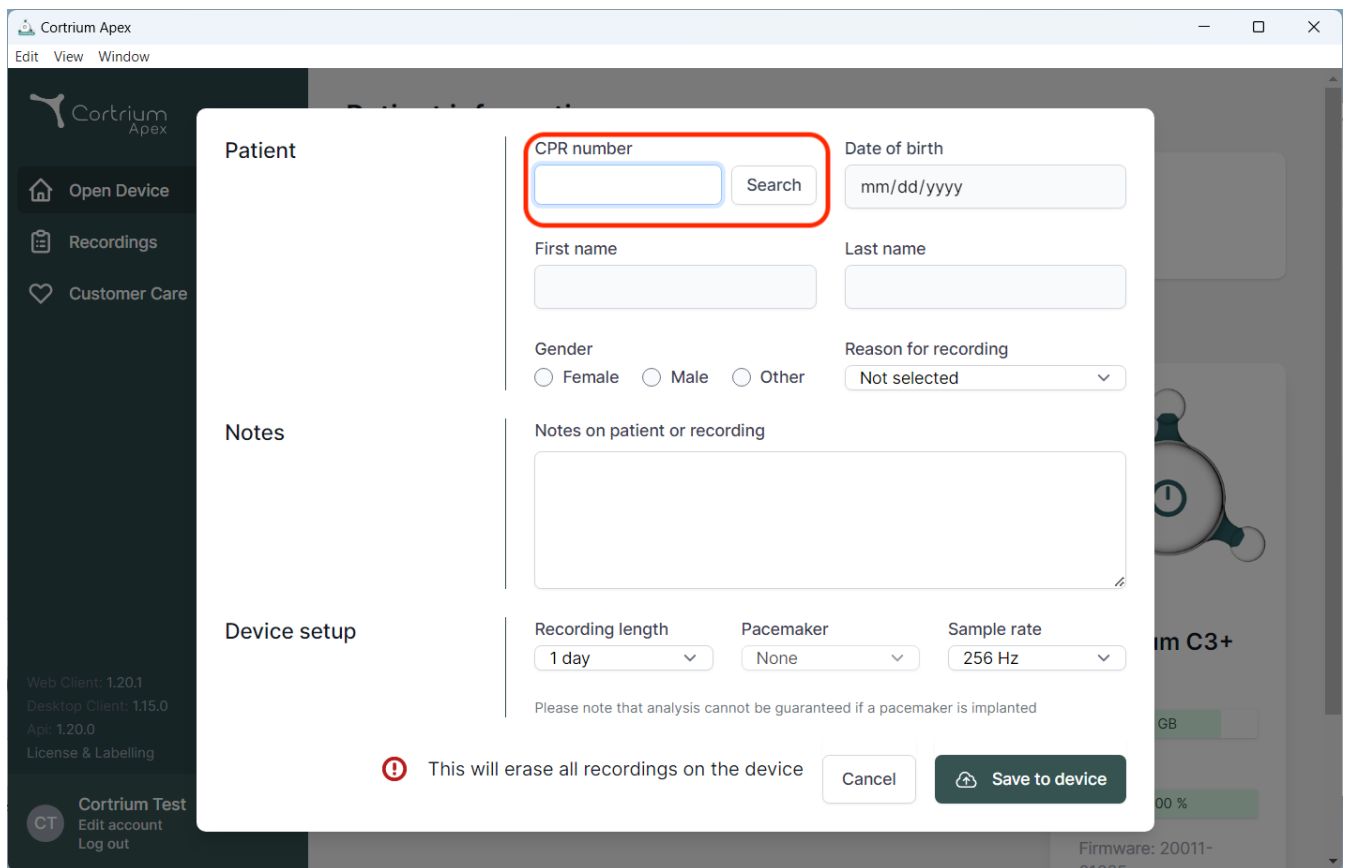
Save changes

Web Client: 1.23.0
Desktop Client: 1.23.0
Api: 1.23.0
License & Labelling

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
The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for searching and adding patient information. The form includes the following fields and controls:

- CPR number:** A text input field with a red border, followed by a "Search" button.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** A section containing three dropdown menus: "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup section, there is a warning message: "⚠ This will erase all recordings on the device". At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

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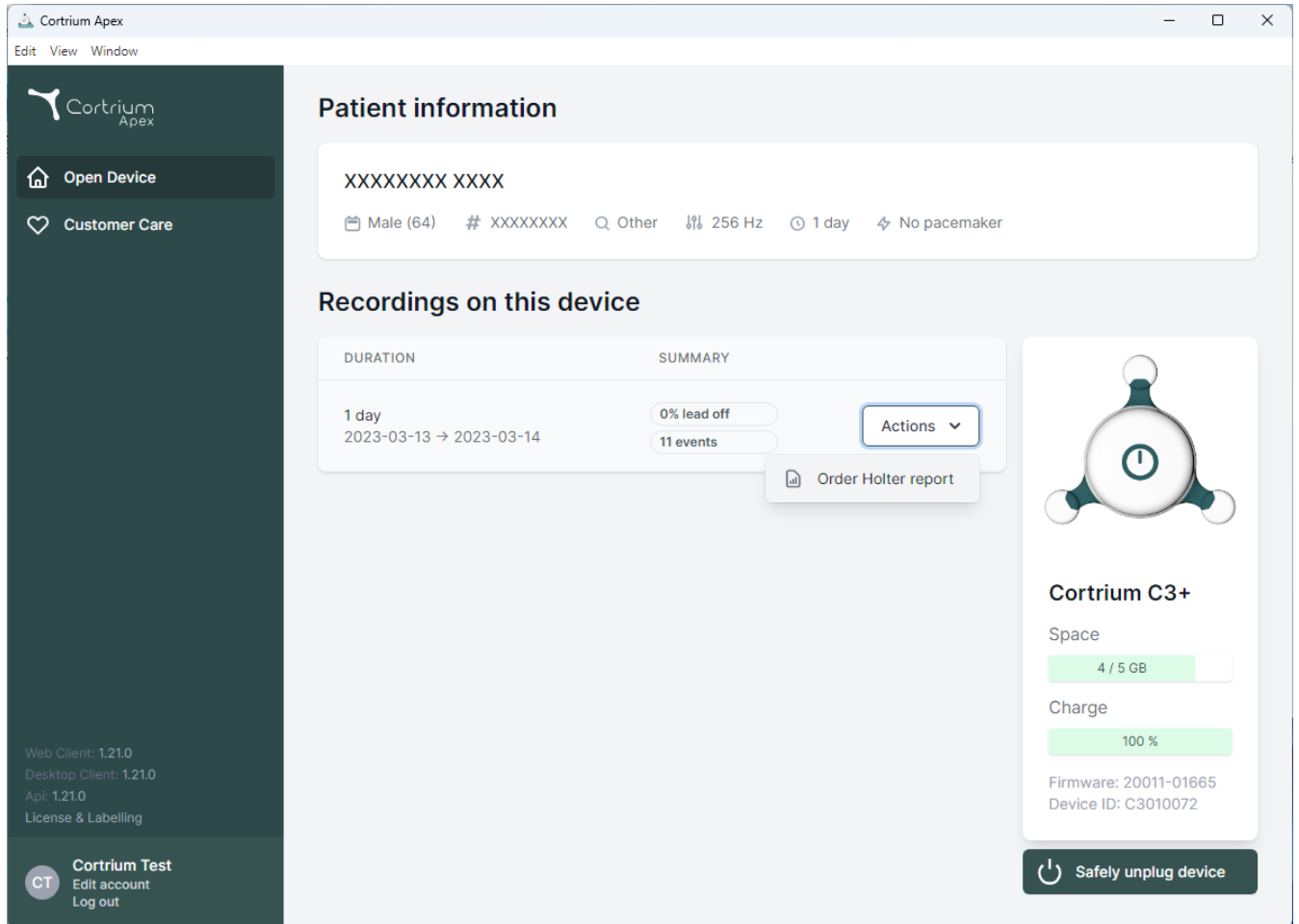The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains the Cortrium Apex logo, 'Open Device', and 'Customer Care' links. The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

Patient information

XXXXXXX XXXX

Male (64) # XXXXXXXX Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

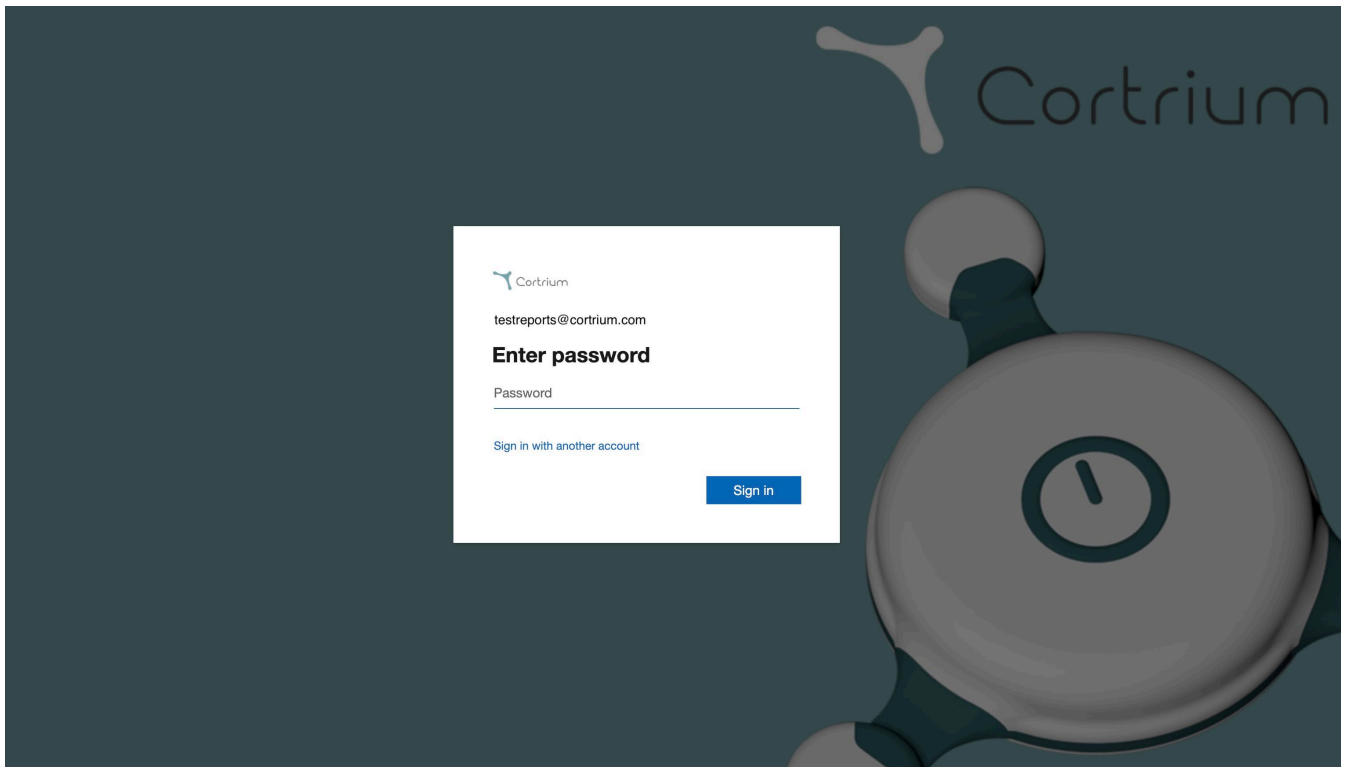
Safely unplug device

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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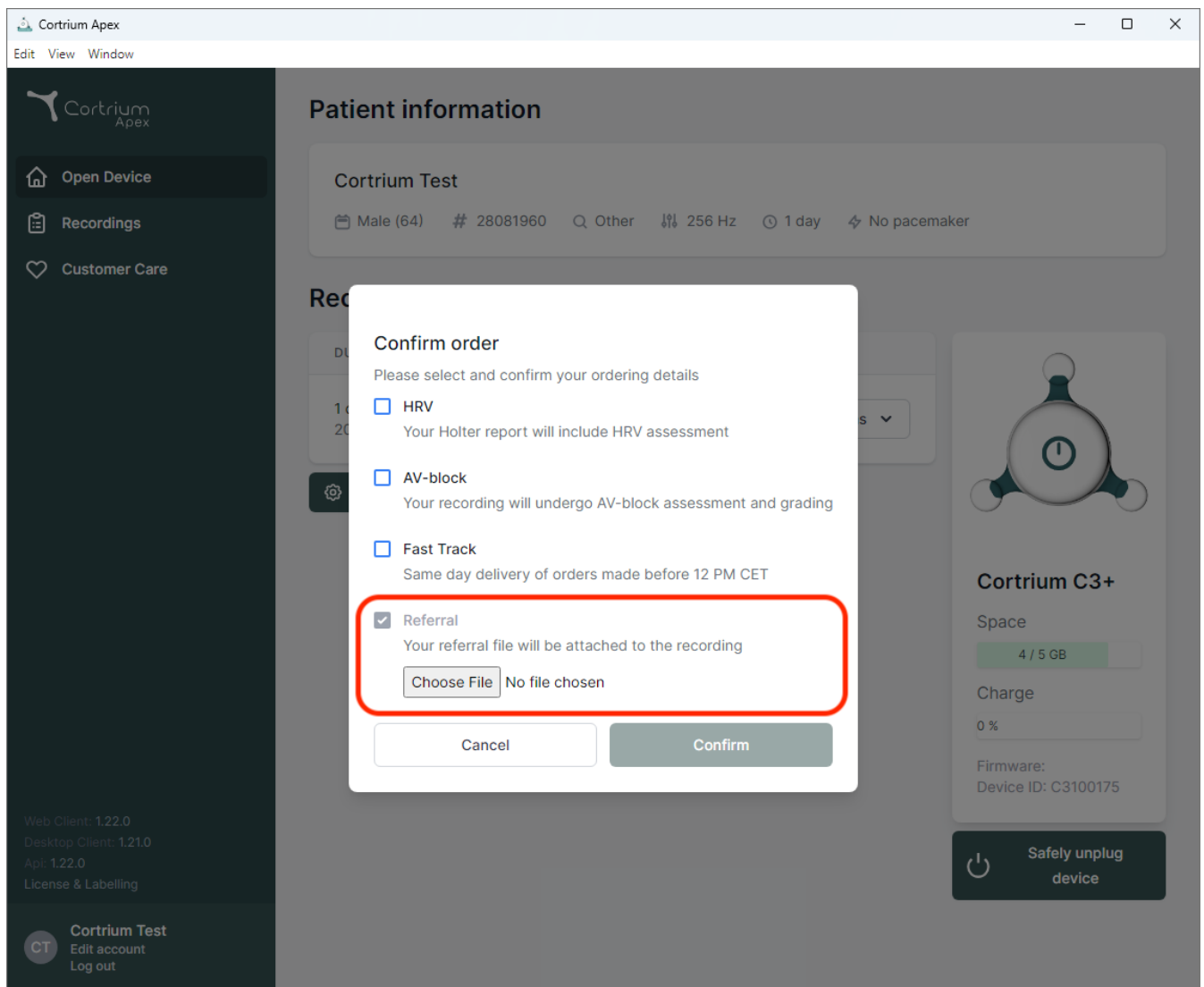
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
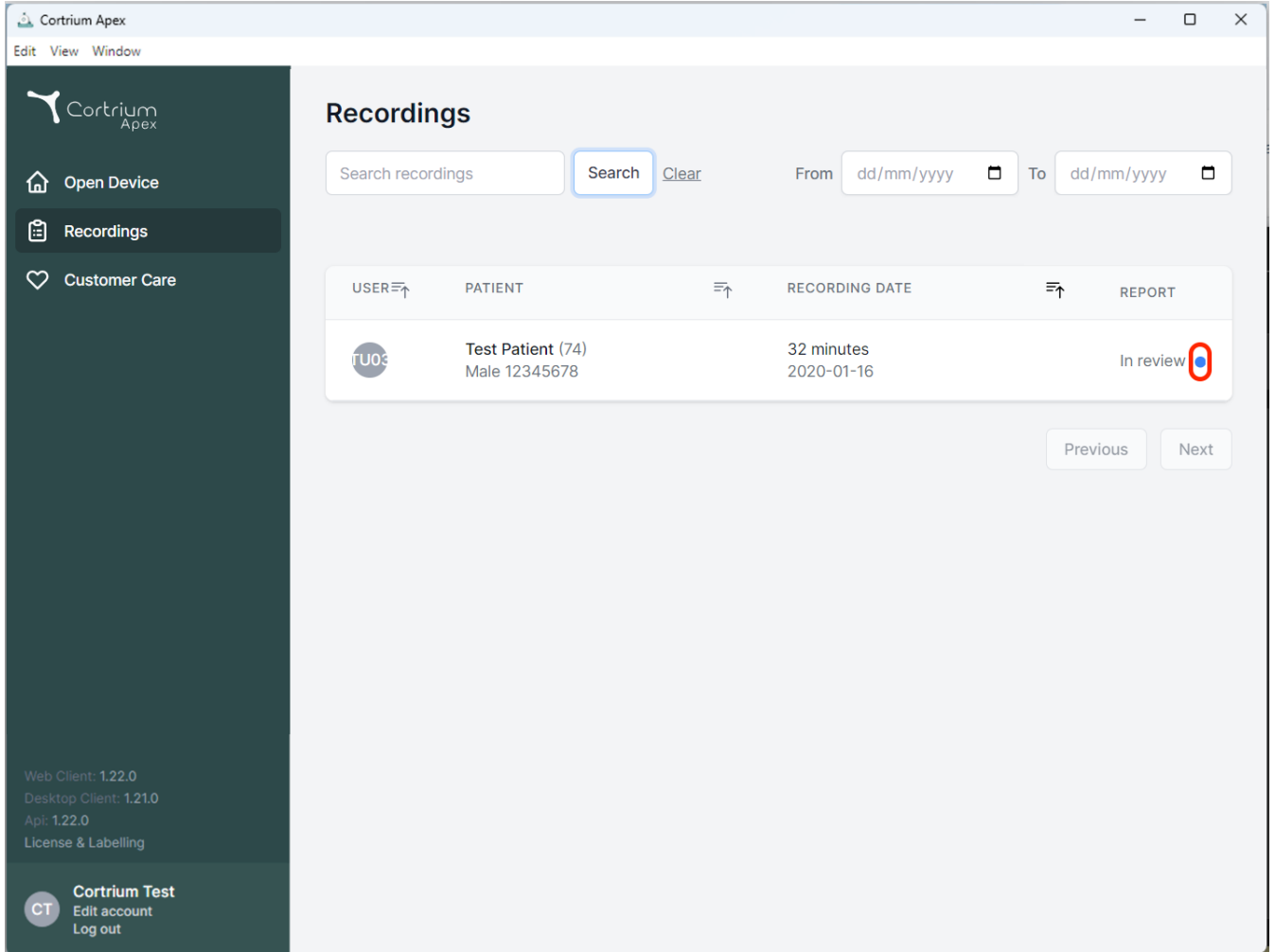
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation links: "Open Device", "Recordings" (selected), and "Customer Care". The main content area is titled "Recordings" and features a search bar with a "Search" button and a "Clear" link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for "Test Patient (74)" with a recording date of "2020-01-16" and a status of "In review". A blue dot is visible next to the "In review" status, indicating a pending report. The bottom of the sidebar shows version information and a user profile for "Cortrium Test".

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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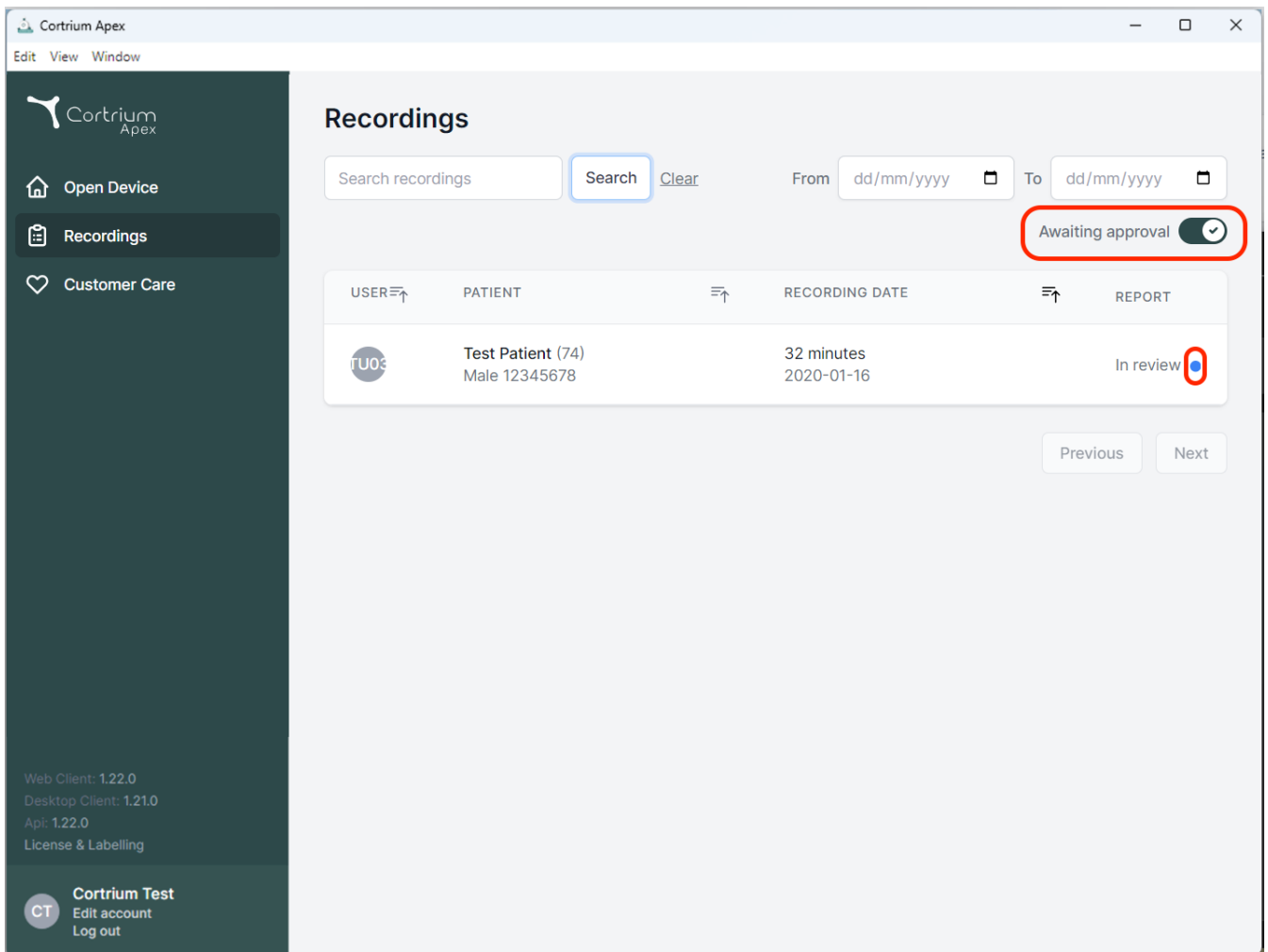
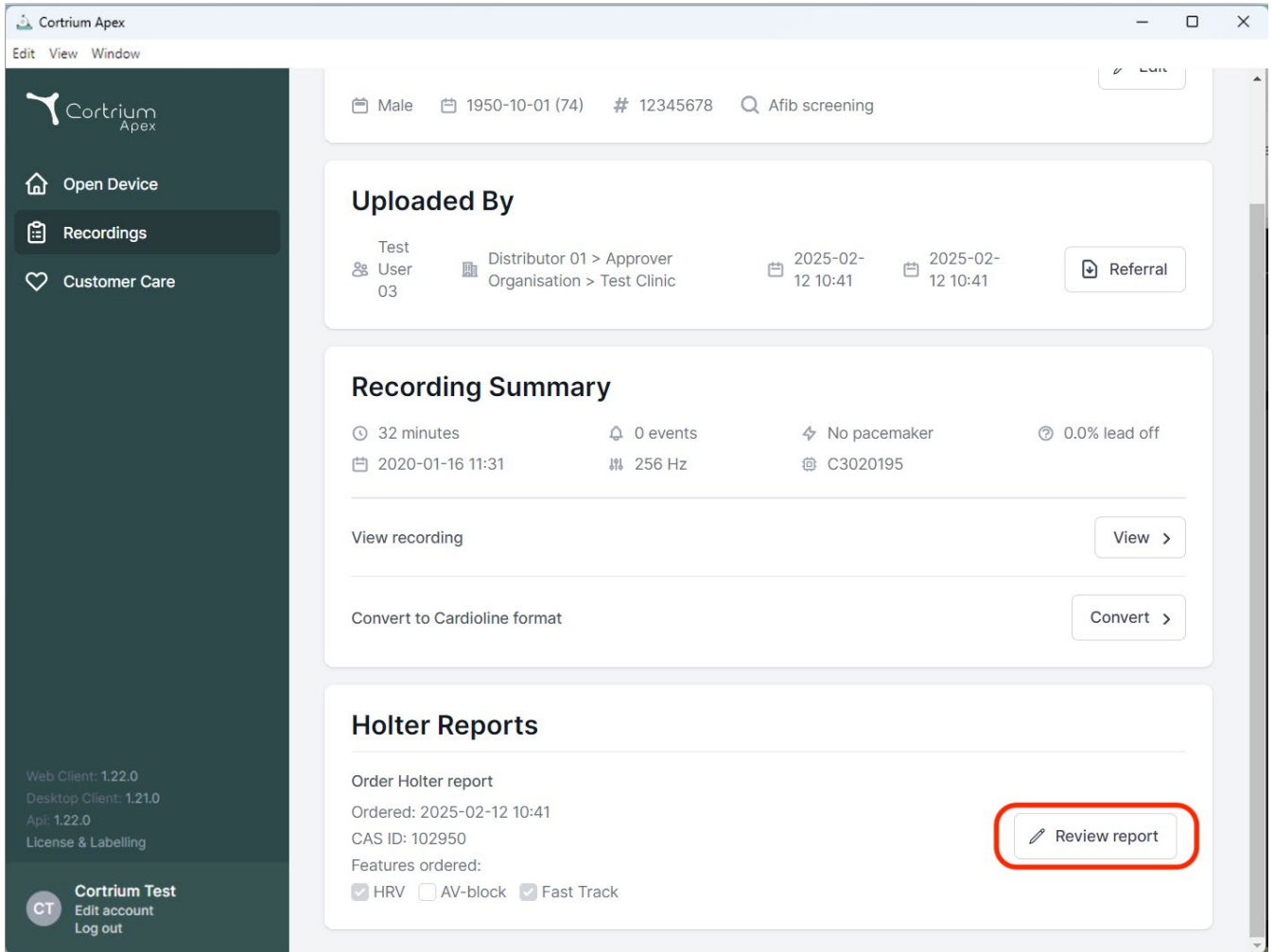
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links for 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into several sections:

- Header Information:** Patient details including 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'.
- Uploaded By:** A section showing the test user (User 03), distributor (Distributor 01 > Approver Organisation > Test Clinic), and upload dates (2025-02-12 10:41). A 'Referral' button is present.
- Recording Summary:** A section providing key recording metrics:
 - 32 minutes
 - 0 events
 - No pacemaker
 - 0.0% lead off
 - 2020-01-16 11:31
 - 256 Hz
 - C3020195
 Below this summary are buttons for 'View recording' and 'Convert to Cardioline format'.
- Holter Reports:** A section for managing holter reports. It includes:
 - 'Order Holter report' button
 - 'Ordered: 2025-02-12 10:41'
 - 'CAS ID: 102950'
 - 'Features ordered:' section with checkboxes for 'HRV' (checked), 'AV-block' (unchecked), and 'Fast Track' (checked).
 - A 'Review report' button, which is highlighted with a red rectangle in the image.

The bottom left corner of the interface shows version information: 'Web Client: 1.22.0', 'Desktop Client: 1.21.0', 'Api: 1.22.0', and 'License & Labelling'. A 'Cortrium Test' section includes links for 'Edit account' and 'Log out'.

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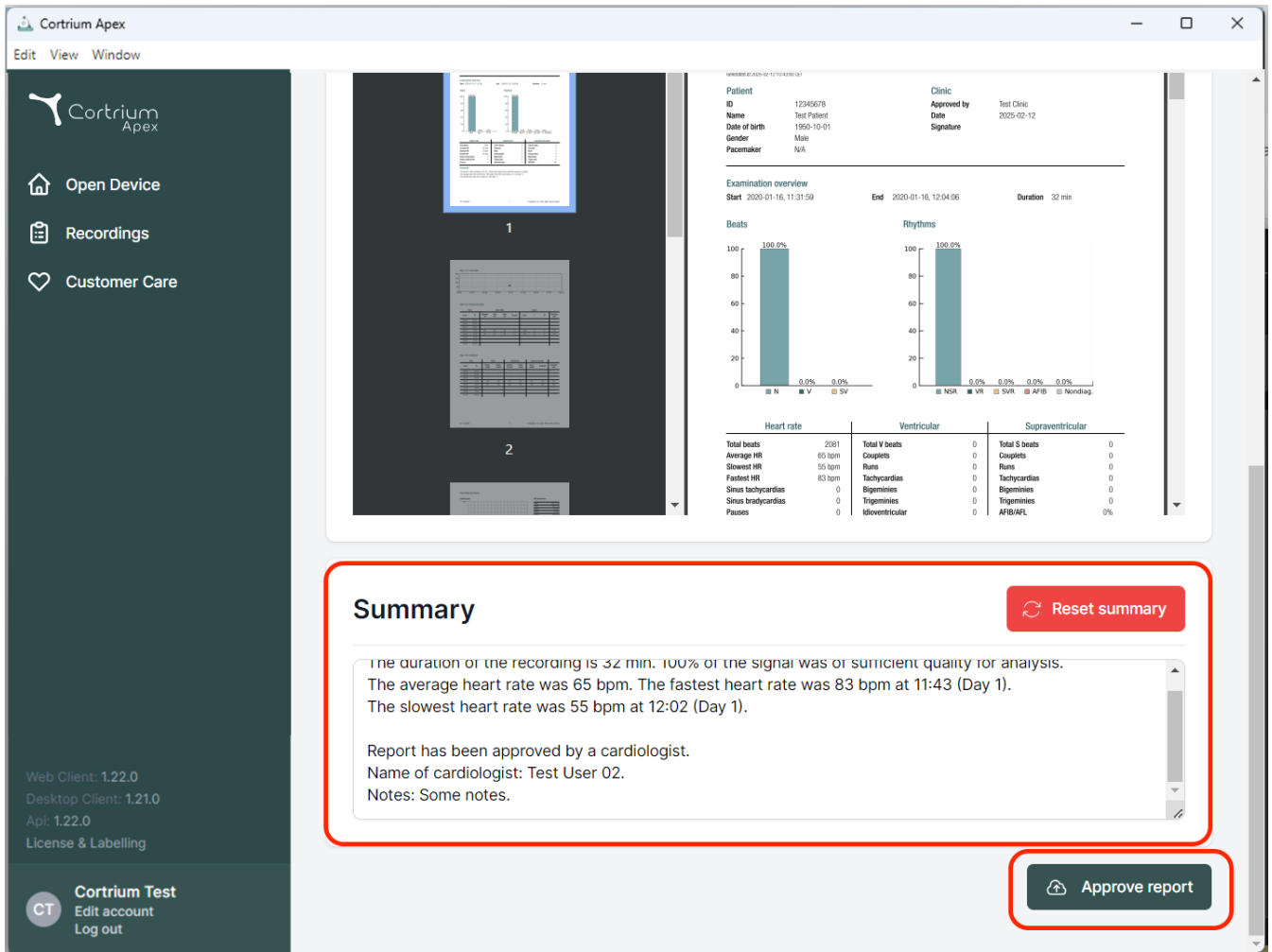Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.22.0
Desktop Client: 1.21.0
Api: 1.22.0
License & Labelling

Cortrium Test
Edit account
Log out

Summary [Reset summary](#)

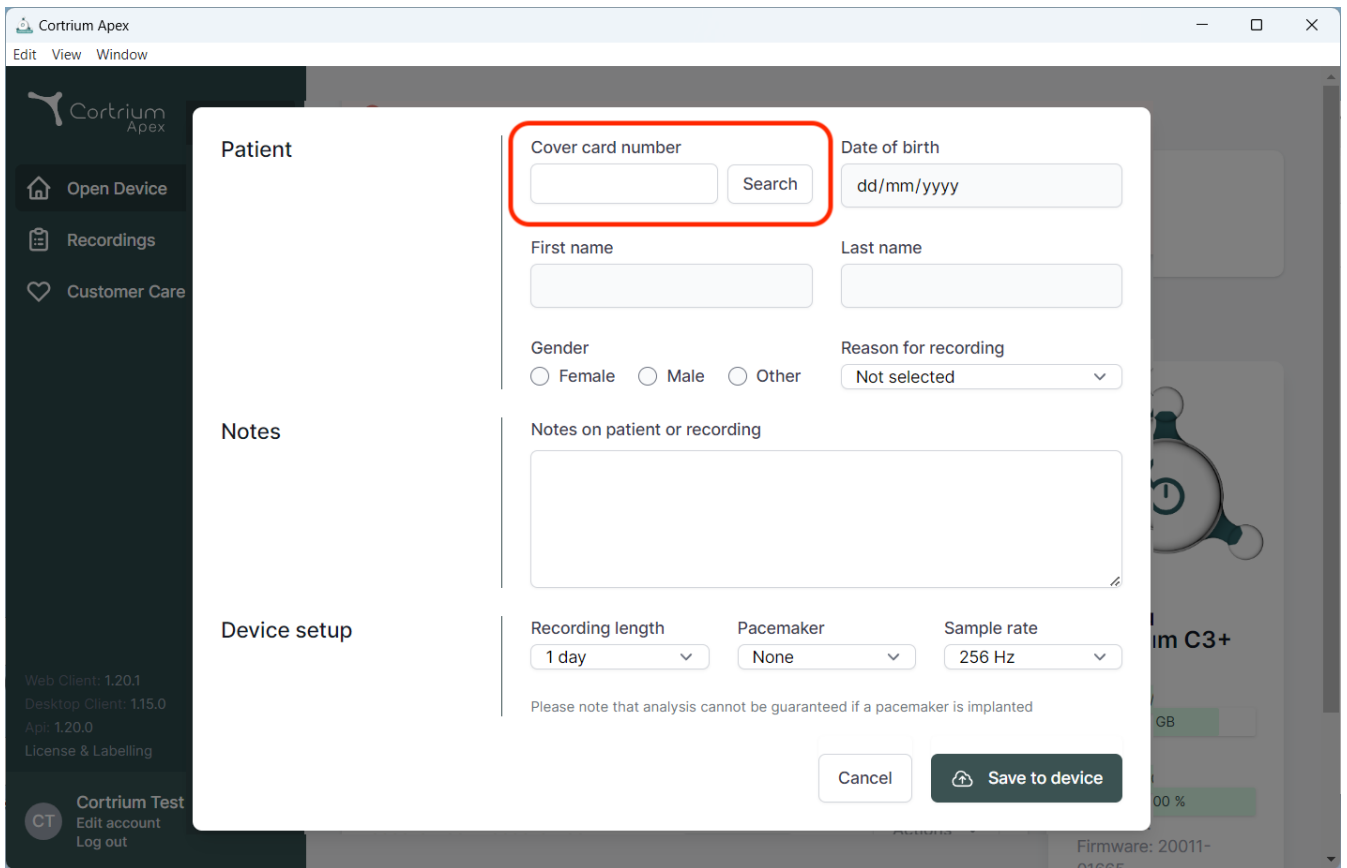
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

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The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying various fields for patient information. The "Cover card number" field is highlighted with a red rectangle. The "Search" button is next to it. Other fields include "Date of birth" (dd/mm/yyyy), "First name", "Last name", "Gender" (Female, Male, Other), "Reason for recording" (Not selected), and "Notes on patient or recording". The "Device setup" section includes "Recording length" (1 day), "Pacemaker" (None), and "Sample rate" (256 Hz). A note at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". Buttons for "Cancel" and "Save to device" are at the bottom right.

Patient

Cover card number Search

Date of birth

First name

Last name

Gender ☐ Female ☐ Male ☐ Other

Reason for recording

Notes

Notes on patient or recording

Device setup

Recording length

Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

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