Cortrium Apex 1.31

Uputstvo za upotrebu

1. Predviđena upotreba i upozorenja	2
2. Instalacija	3
3. Prijavite se na Cortrium Apex	5
3.1 Držite korisnike prijavljene na lokalnom računaru u Cortrium Apex-u	6
4. Pripremite C3w/C3 ⁺ za novog pacijenta	8
5. Naručite holter izveštaj	12
6. Pregled holter izveštaja i celog snimka	16
6.1 Pregled holter izveštaja	16
6.2 Pregledač snimaka	19
6.3 Uređivač sažetka	21
7. Uredite detalje o pacijentu i pogledajte detalje snimaka	21
8. Brisanje snimka	23
9. Preuzmite EDF datoteku	25
9.1 Način preuzimanja EDF datoteke pre naručivanja holter izveštaja	25
9.2 Način preuzimanja EDF datoteke nakon naručivanja izveštaja	27
10. Odjava iz Cortrium Apex-a	28
11. Ažuriranja softvera	28
12. Minimalni tehnički zahtevi	28
13. Rešavanje problema	29

13.1 „Čini se da je uređaj podešen za pacijenta...“	29
13.2 Na sažetku snimanja prikazuje se „n/p odvojena elektroda“ ili „n/p događaji“	29
13.3 Internet veza je prekinuta.	29
13.4 Apex je novoinstaliran, ali se ne otvara	30
14. Bezbednosne informacije	30
15. Obeležavanje i simboli	31
16. Izjava o infrastrukturi	32
Prilog I	33
i. Dostupne integracije	33
ii. GDT datoteka GDT	34
iii. DICOM inkapsulirani izveštaj.....	35
iv. Traženje pacijenta po CPR-u	37
v. Uloga samo otpremaoca	38
vi. Integracija aktivnog direktorijuma	39
vii. Upućivanje pacijenata.....	40
viii. Tok odobravanja izveštaja	43
ix. Pretraga Nereida pacijenta i isporuka izveštaja	46
x. Spisak podržanih uređaja	47

1. Predviđena upotreba i upozorenja

Namena

Cortrium Apex je softverska aplikacija namenjena za podešavanje, konfiguraciju, prenos i skladištenje EKG podataka sa podržanih Cortrium EKG snimača. Namenjen je za upotrebu od strane zdravstvenih radnika u klinikama, bolnicama i okruženjima daljinske nege za upravljanje podacima o pacijentima, naručivanje i pregled izveštaja o EKG analizi i pregled kompletnih EKG snimaka. Cortrium Apex podržava naručivanje

EKG analize putem Cortrium-ove usluge analize (CAS) i takođe omogućava preuzimanje i izvoz EKG podataka u softver drugih proizvođača radi dalje analize.

Predviđeni korisnici

Softver je namenjen da ga koriste zdravstveni radnici u privatnoj praksi, klinikama i bolnicama, kao i na daljinu.

Upozorenja

- Ovaj proizvod se može koristiti samo u skladu sa njegovom predviđenom upotrebom kako je navedeno u ovom uputstvu
- Korisnici ovog proizvoda su jedinstveni i nikada ne bi trebalo da dele lozinke
- Snimci ekrana softvera mogu da sadrže osetljive informacije i treba ih izbegavati
- Podaci koji se dele van sistema su na sopstveni rizik korisnika
- U slučaju ozbiljnih incidenata u vezi sa softverom, obratite se lokalnom nadležnom organu, kao i kompaniji Cortrium putem e-pošte: customercare@cortrium.com bez nepotrebnog odlaganja.

Dodatne informacije

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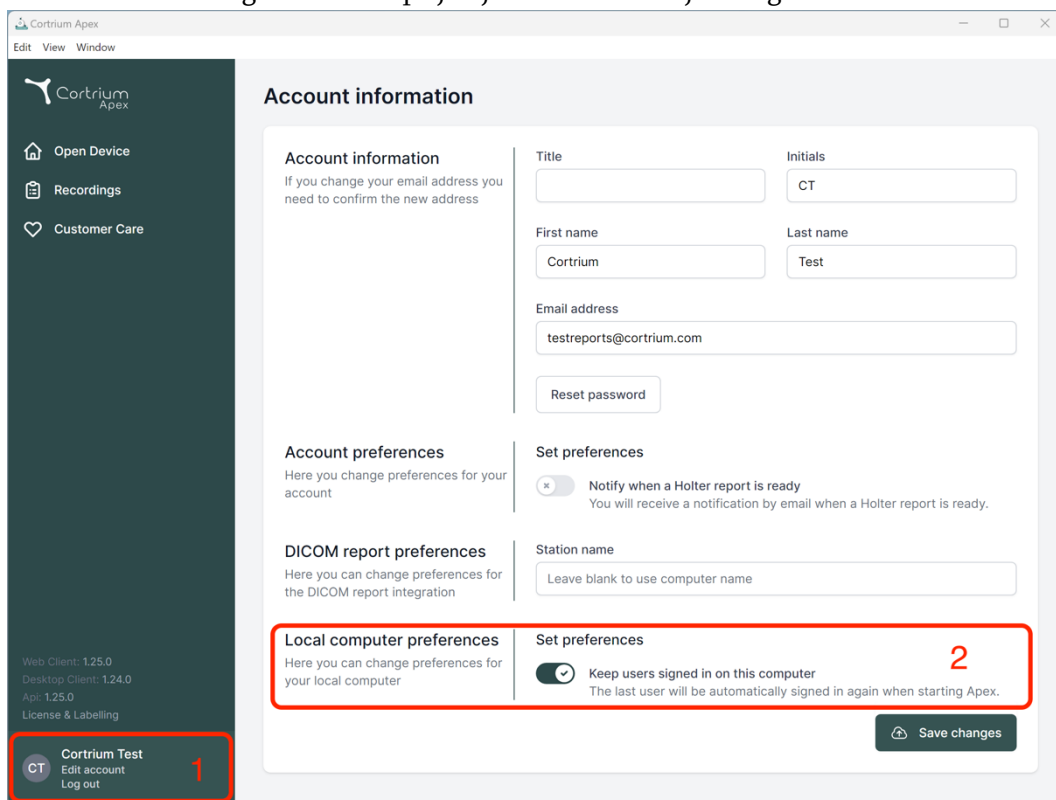
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client, Desktop Client, and API, along with a 'License & Labelling' link. At the bottom of the sidebar, a user profile for 'Cortrium Test' is shown with options to 'Edit account' and 'Log out'. A red box labeled '1' highlights this user profile section.

The main content area is titled 'Account information'. It contains several sections:

- Account information:** Fields for Title, Initials (CT), First name (Cortrium), Last name (Test), and Email address (testreports@cortrium.com). A 'Reset password' button is also present.
- Account preferences:** A section for changing account preferences.
- DICOM report preferences:** A section for changing preferences for DICOM report integration.
- Local computer preferences:** A section for changing preferences for the local computer. A red box labeled '2' highlights the 'Keep users signed in on this computer' toggle, which is currently turned on. Below this toggle, it states: 'The last user will be automatically signed in again when starting Apex.'
- Set preferences:** A section for setting preferences, including a 'Station name' field.

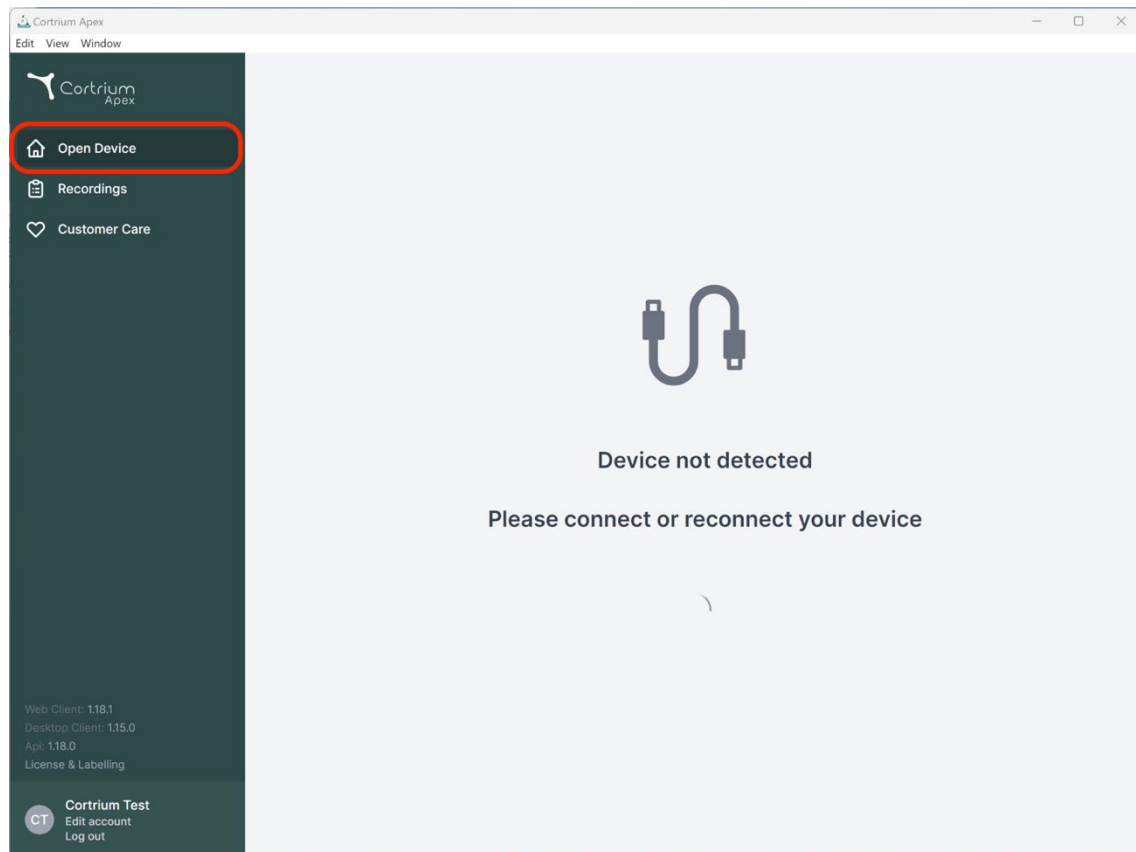
 A 'Save changes' button is located at the bottom right of the main content area.

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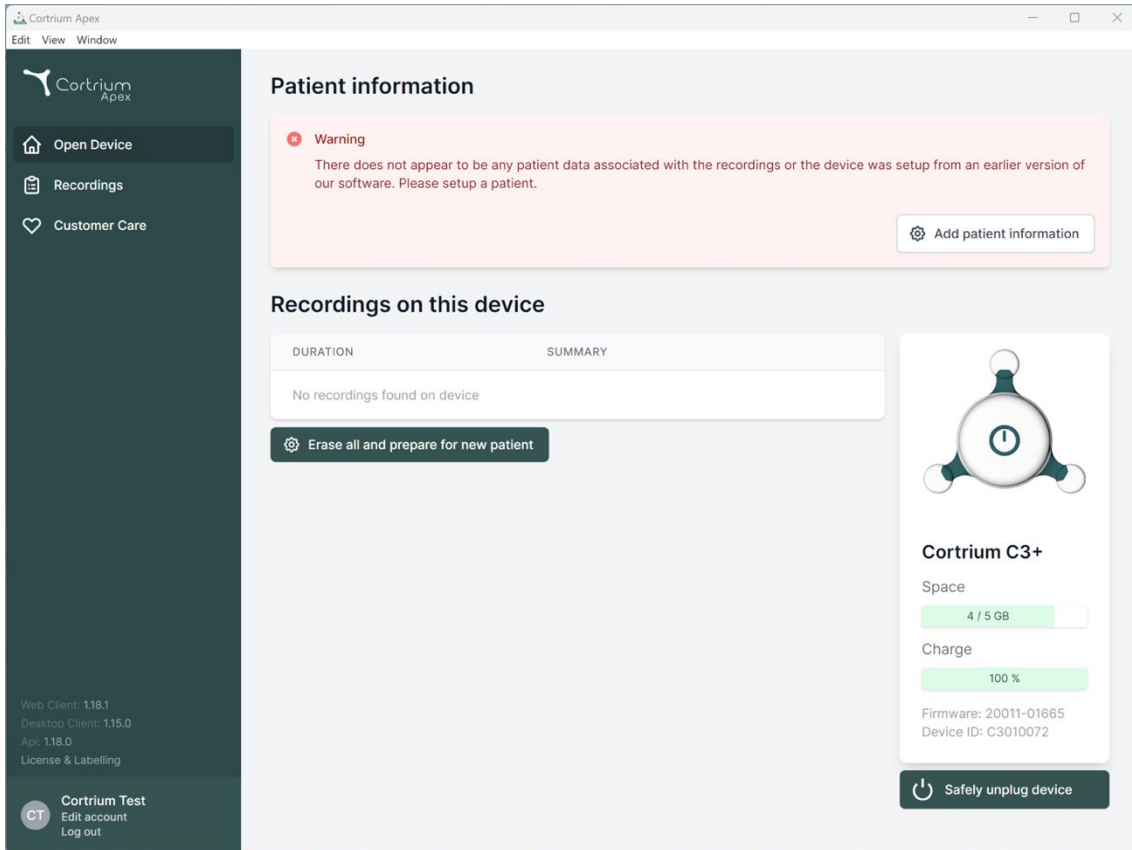
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area has a 'Patient information' section with a warning message: 'There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' and an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table showing 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right is a device status card for 'Cortrium C3+' showing 'Space' (4 / 5 GB) and 'Charge' (100 %) with progress bars, and a 'Safely unplug device' button at the bottom.

- Cortrium Apex prikazuje informacije o povezanom uređaju i svim podacima sačuvanim na njemu.
- Izaberite „Izbriši sve i pripremi uređaj za novog pacijenta“.
- Popunite relevantne informacije o pacijentu i evidenciju:

Cortrium Apex
Edit View Window

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

Save to device

Warning This will erase all recordings on the device

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
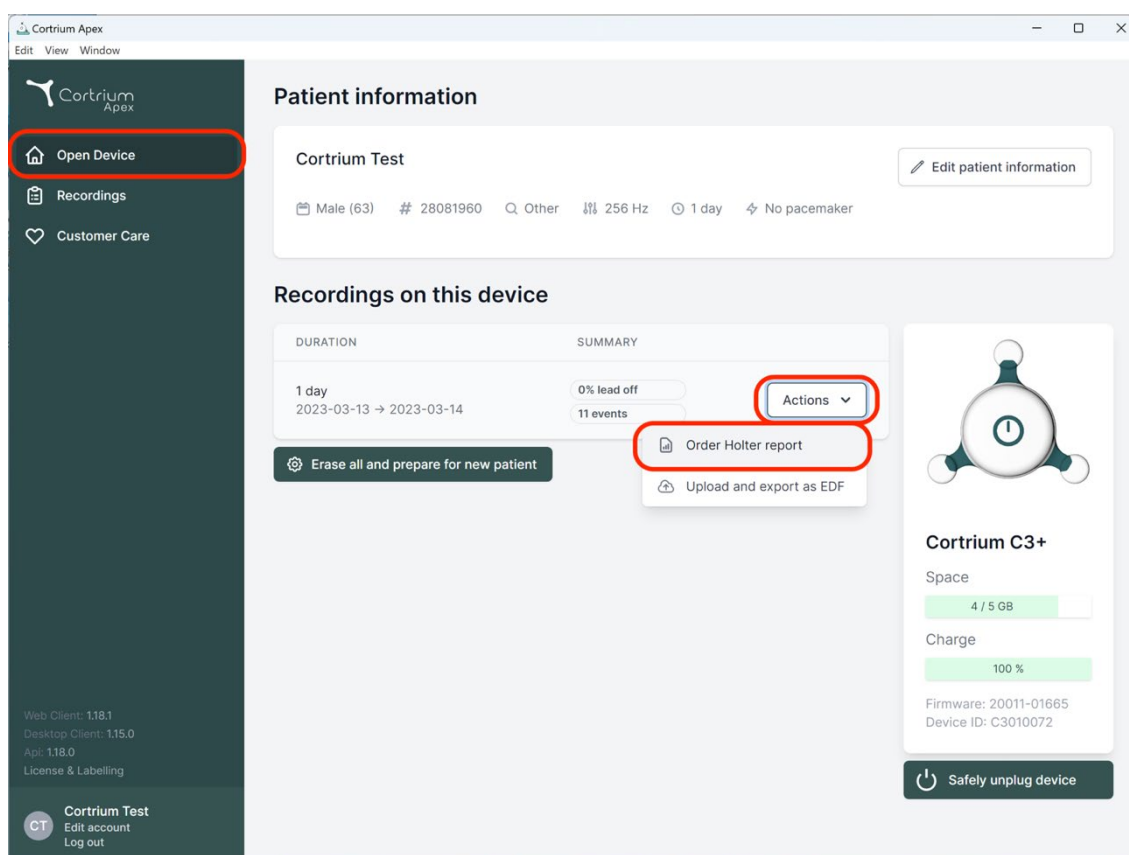
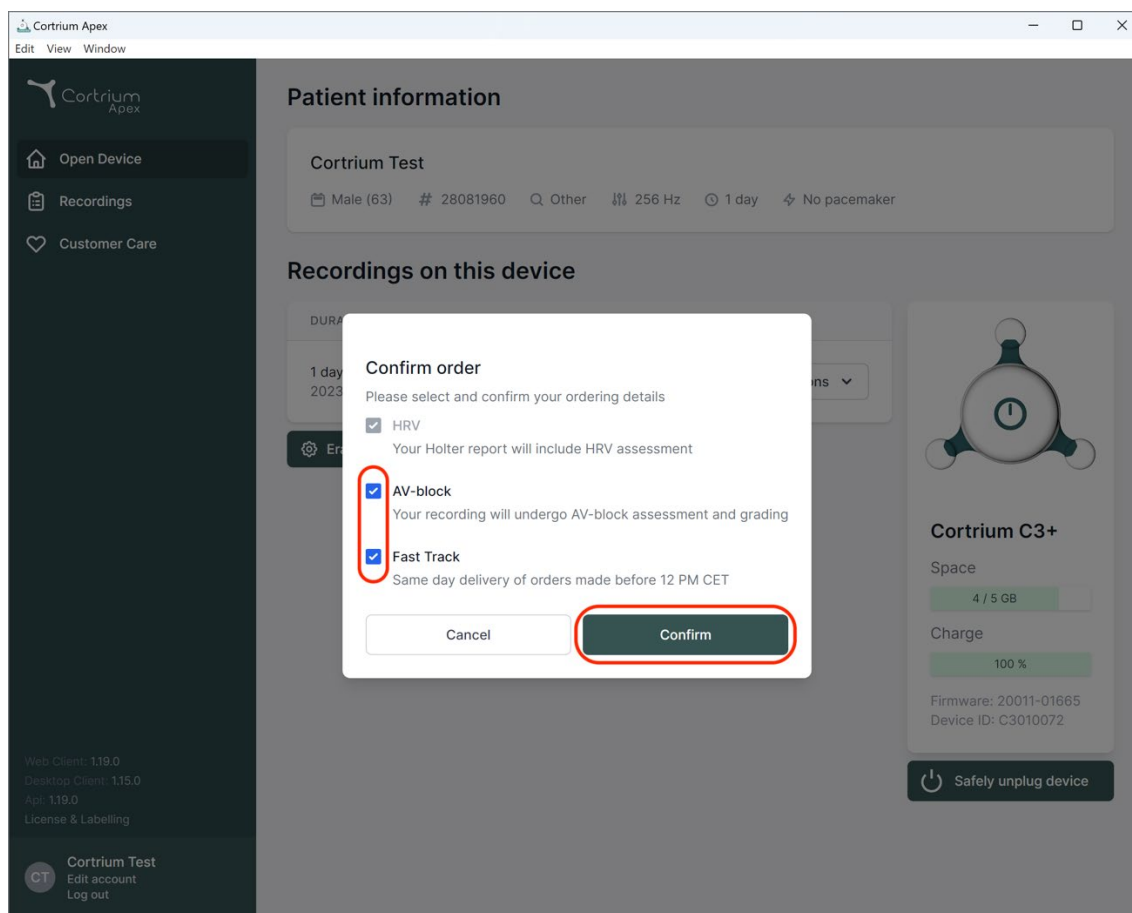
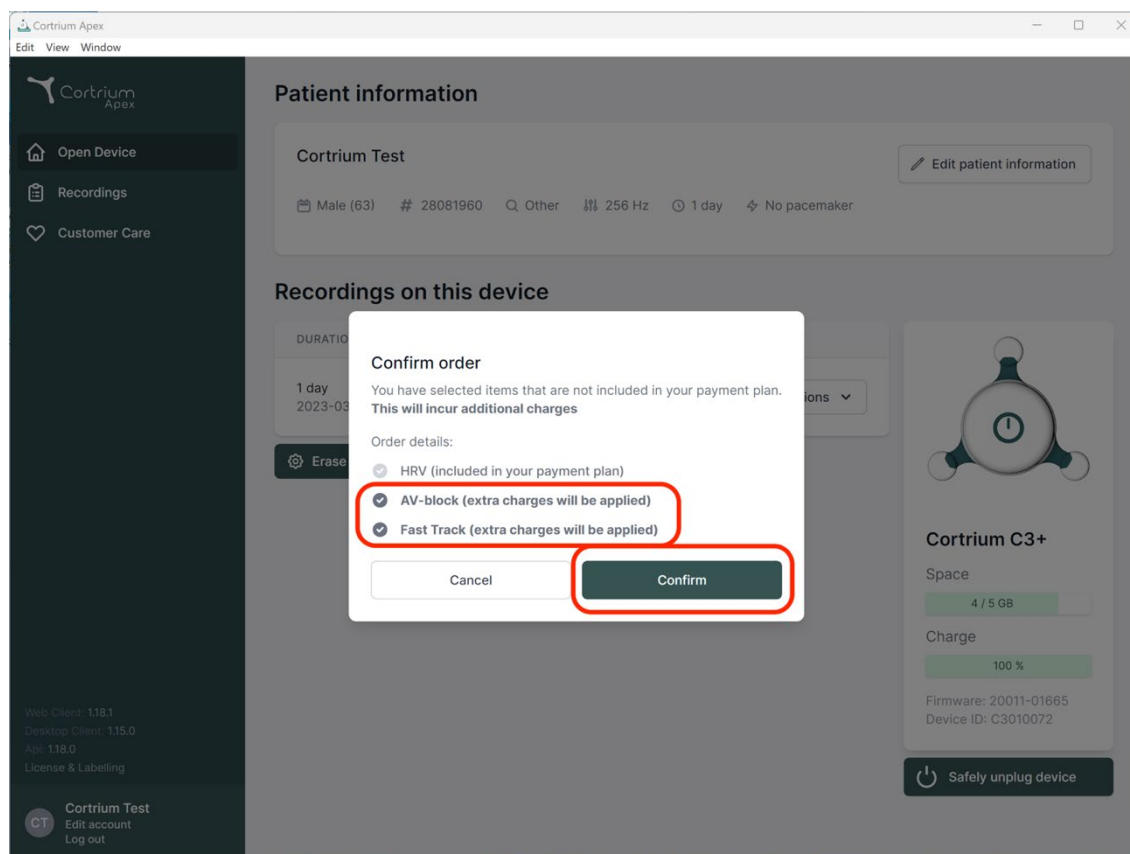
Da biste nastavili, kliknite na „Potvrdi“.

Snimak se sada otprema.

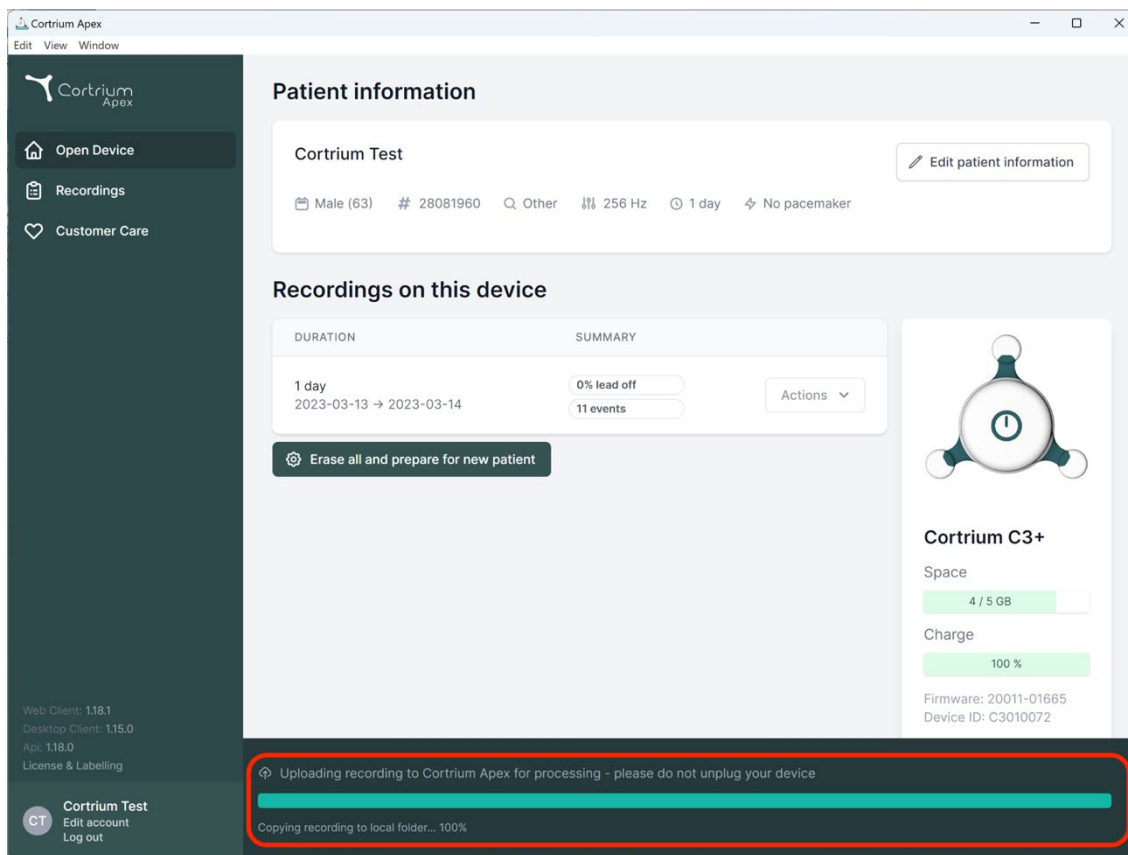The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: Open Device, Recordings, and Customer Care. Below these are system status indicators: Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, and License & Labelling. At the bottom of the sidebar is a user profile for 'Cortrium Test' with options to edit the account or log out.

The main content area is titled 'Patient information' and displays data for 'Cortrium Test'. It includes fields for gender (Male, 63), ID (28081960), other settings (Other, 256 Hz, 1 day, No pacemaker), and an 'Edit patient information' button. Below this is a section 'Recordings on this device' with a table showing a 1-day recording from 2023-03-13 to 2023-03-14. The summary shows 0% lead off and 11 events, with an 'Actions' dropdown. A button 'Erase all and prepare for new patient' is also present.

On the right side, there is a device status card for 'Cortrium C3+' showing 'Space' at 4 / 5 GB and 'Charge' at 100%. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072'.

At the bottom, a red-bordered box highlights a status message: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device'. Below this message is a green progress bar and the text 'Copying recording to local folder... 100%'.

Nakon otpremanja snimka, holter izveštaj će biti dostupan kada se analizira. Uređaj se već može pripremiti za novog pacijenta čim se snimak otpremi.

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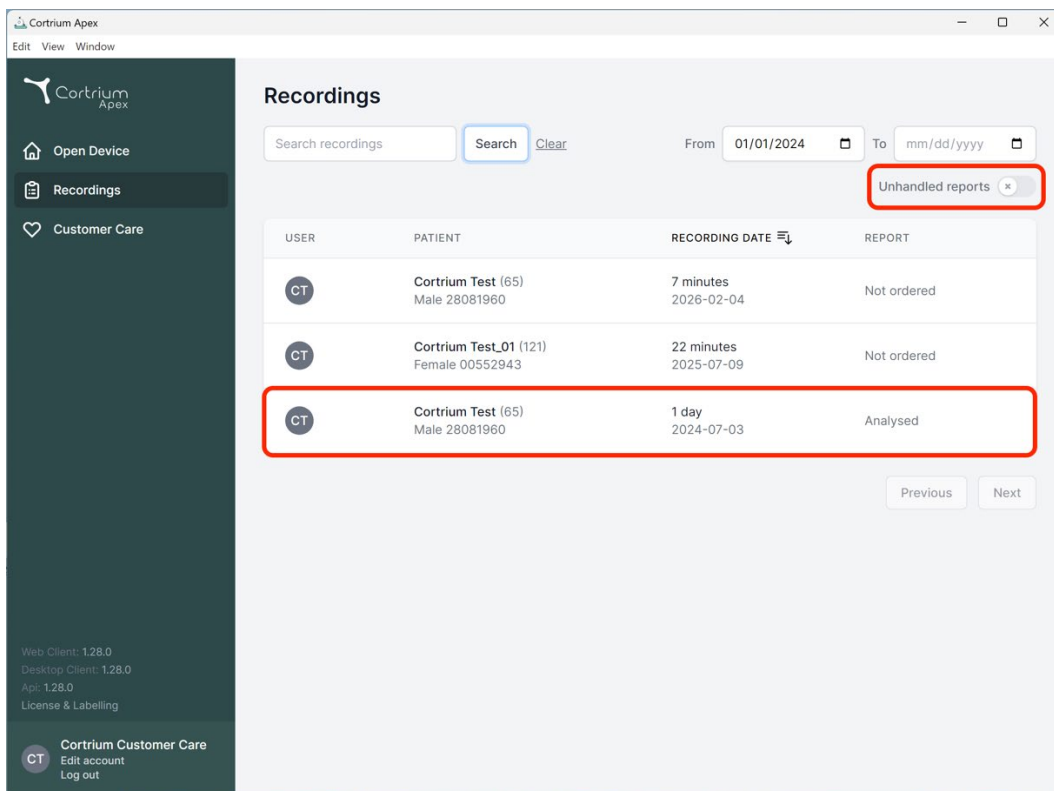
The screenshot displays the 'Recordings' section of the Cortrium Apex interface. The left sidebar contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main area shows a table of recordings with the following data:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (65) Male 28081960	7 minutes 2026-02-04	Not ordered
CT	Cortrium Test_01 (121) Female 00552943	22 minutes 2025-07-09	Not ordered
CT	Cortrium Test (65) Male 28081960	1 day 2024-07-03	Analysed

At the top right of the table, there is a filter button labeled 'Unhandled reports'. Below the table, there are 'Previous' and 'Next' navigation buttons.

Sistem omogućava da snimci budu označeni kao da sadrže kritične ili značajne nalaze. Ovi markeri su prikazani na listi snimaka i namenjeni su za podršku kliničkom određivanju prioriteta tako što omogućavaju bržu identifikaciju snimaka koji mogu zahtevati brz pregled. Kriterijumi koji se koriste za klasifikaciju dostupni su korisniku kada pređete pokazivačem miša preko odgovarajuće ikone zastavice.

Kategorija	Značajni nalazi	Kritični nalazi
Vizuelna ikona		
Pauziraj	Duže od 5 sekundi	Duže od 7 sekundi
Ventrikularna tahikardija	Epizoda/e sa 10 ili više otkucaja	Trajni VT u trajanju od 30 sekundi ili duže
Srčana frekvencija	≥ 200 otk./min. ≤ 30 otk./min.	Prosečna srčana frekvencija ≤ 35 otk./min.
AV-blok	-	3. stepen / Visoki stepen AV-blok (ako je naručeno)

Cortrium Apex

Edit View Window

Open Device

Recordings

Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT

Cortrium Test

Edit account
Log out

Recording details

Delete recording

Cortrium Test

Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording

View >

Download recording in .EDF format

Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

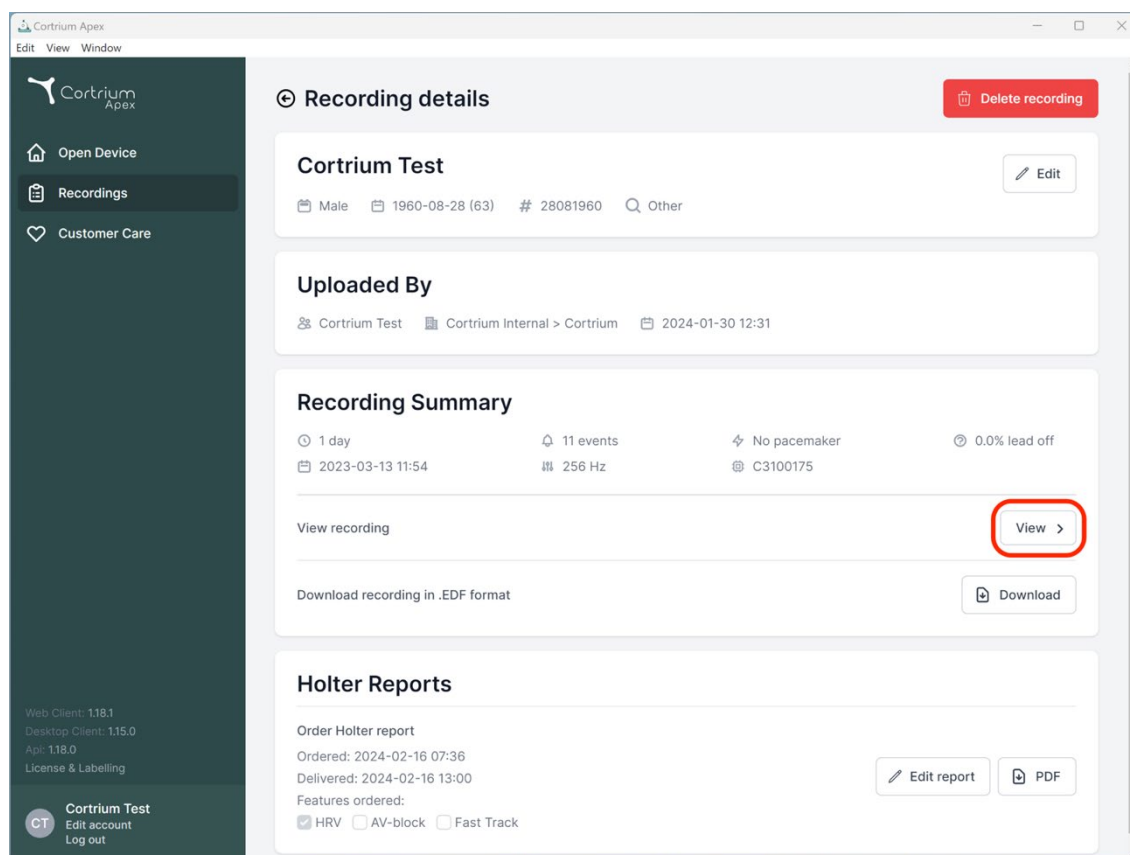
Features ordered:
☒ HRV ☐ AV-block ☐ Fast Track

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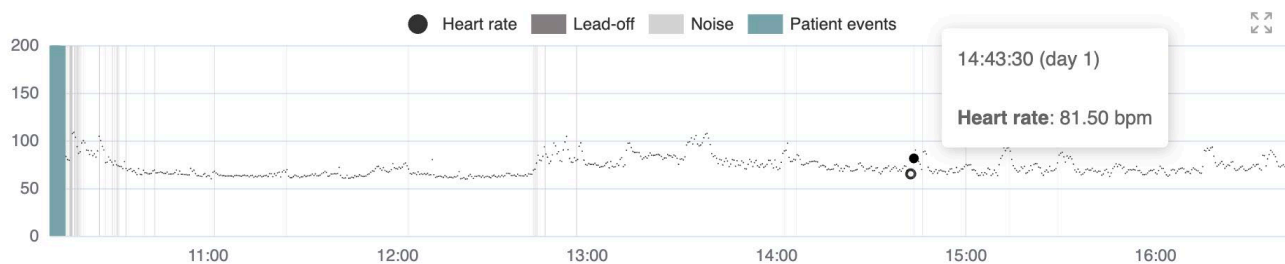
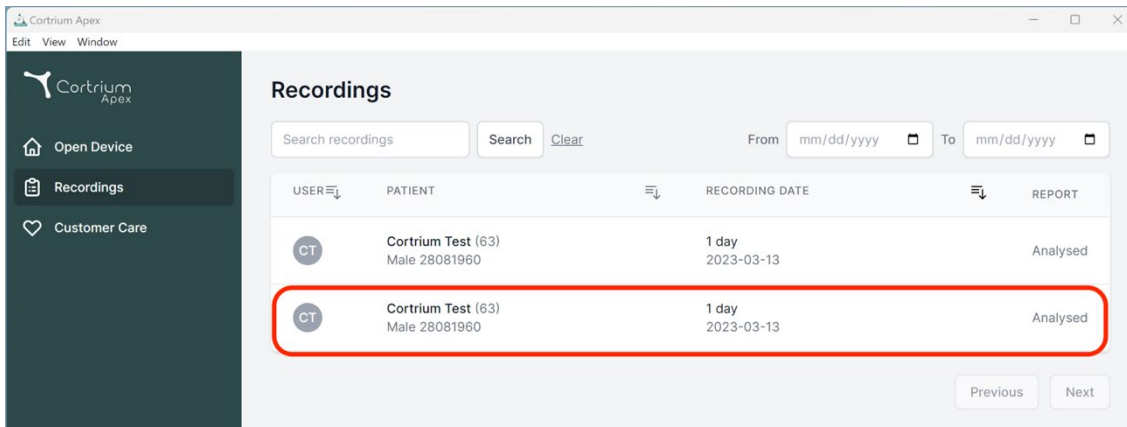
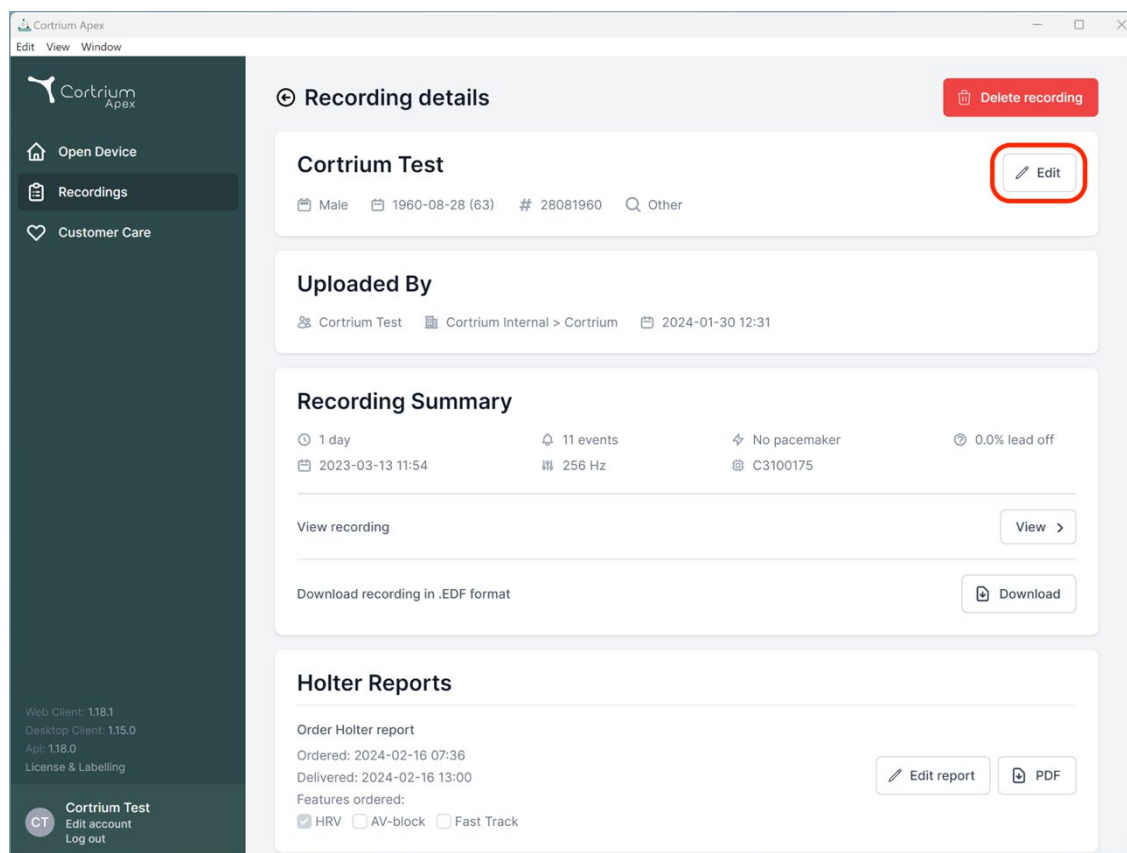
Order Holter report

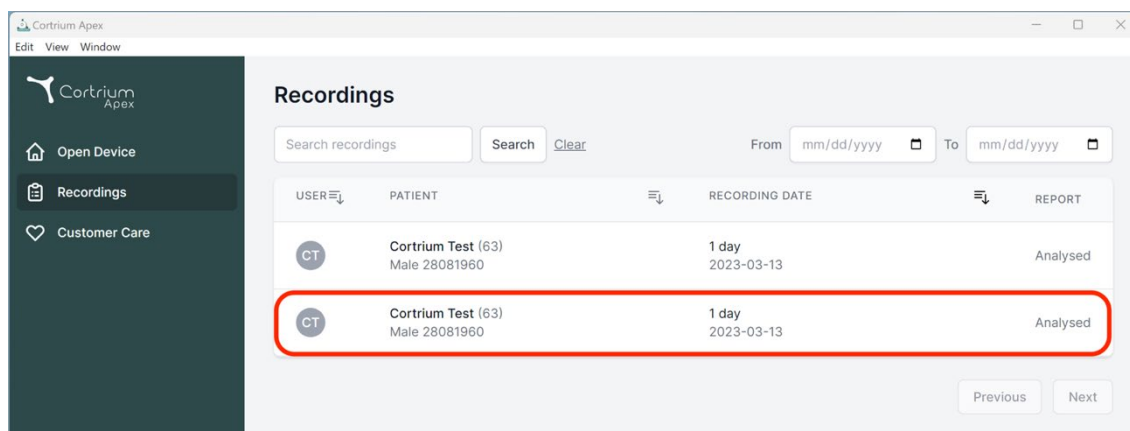
Ordered: 2024-02-16 07:36 Delivered: 2024-02-16 13:00

Features ordered: ☒ HRV ☐ AV-block ☐ Fast Track

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
Recordings

Search recordings Search Clear From mm/dd/yyyy To mm/dd/yyyy

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

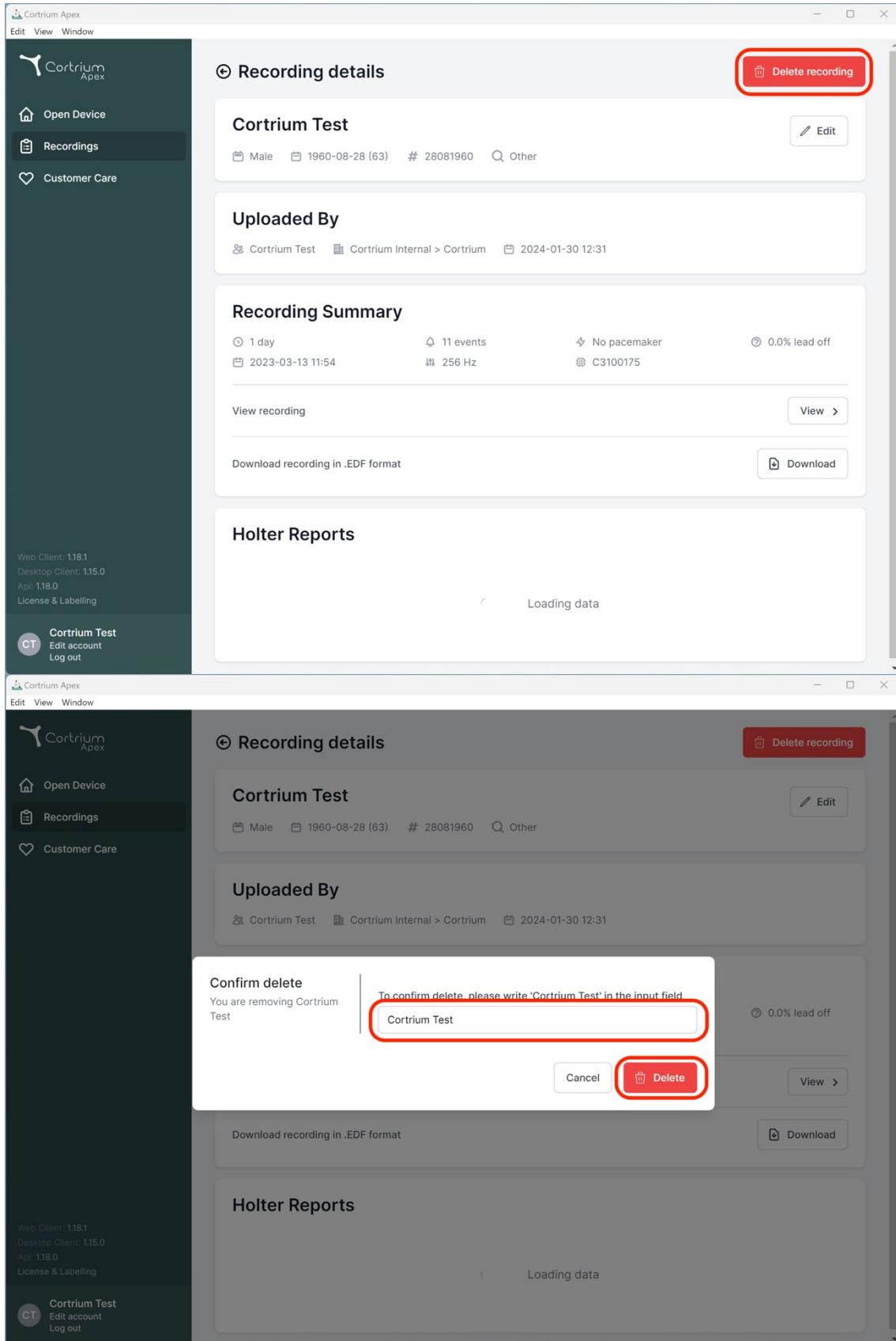Previous Next

Recording details Delete recording

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View recording View >

Download recording in .EDF format Download

Holter Reports

Loading data

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

Cortrium Test

Cancel Delete

Download recording in .EDF format Download

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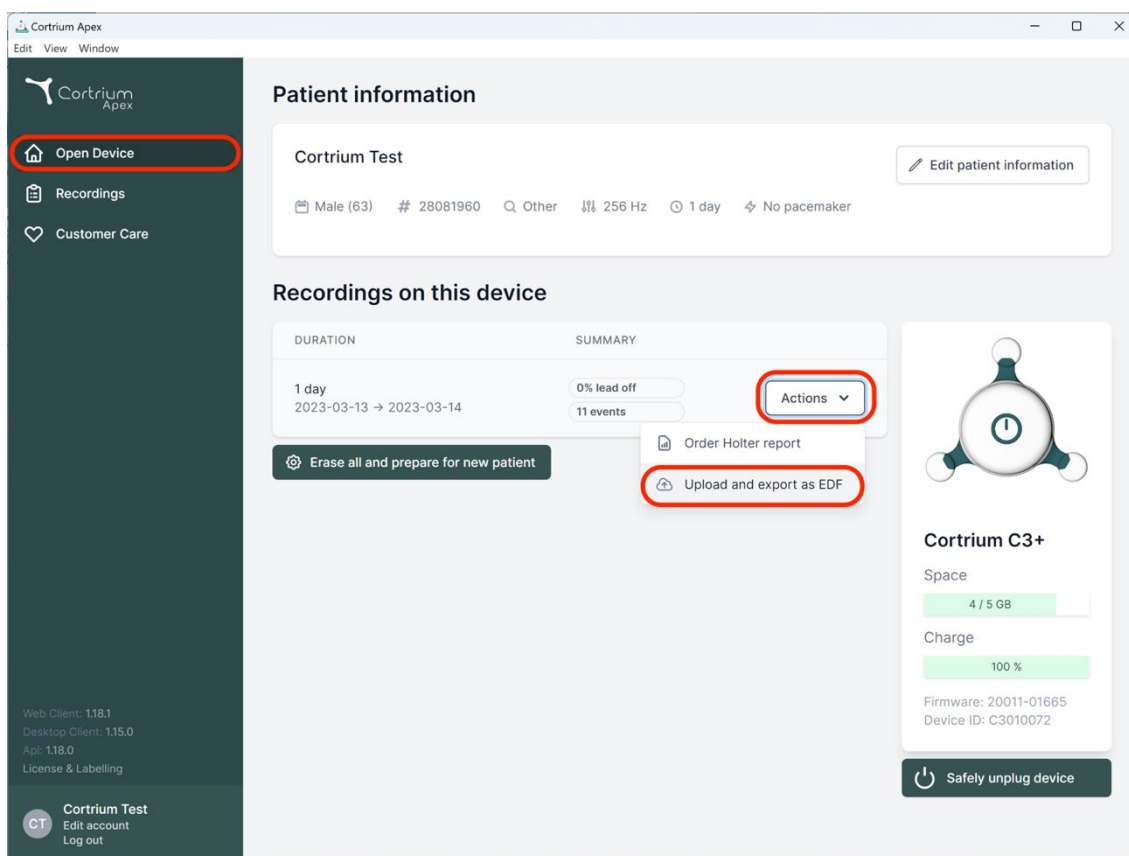
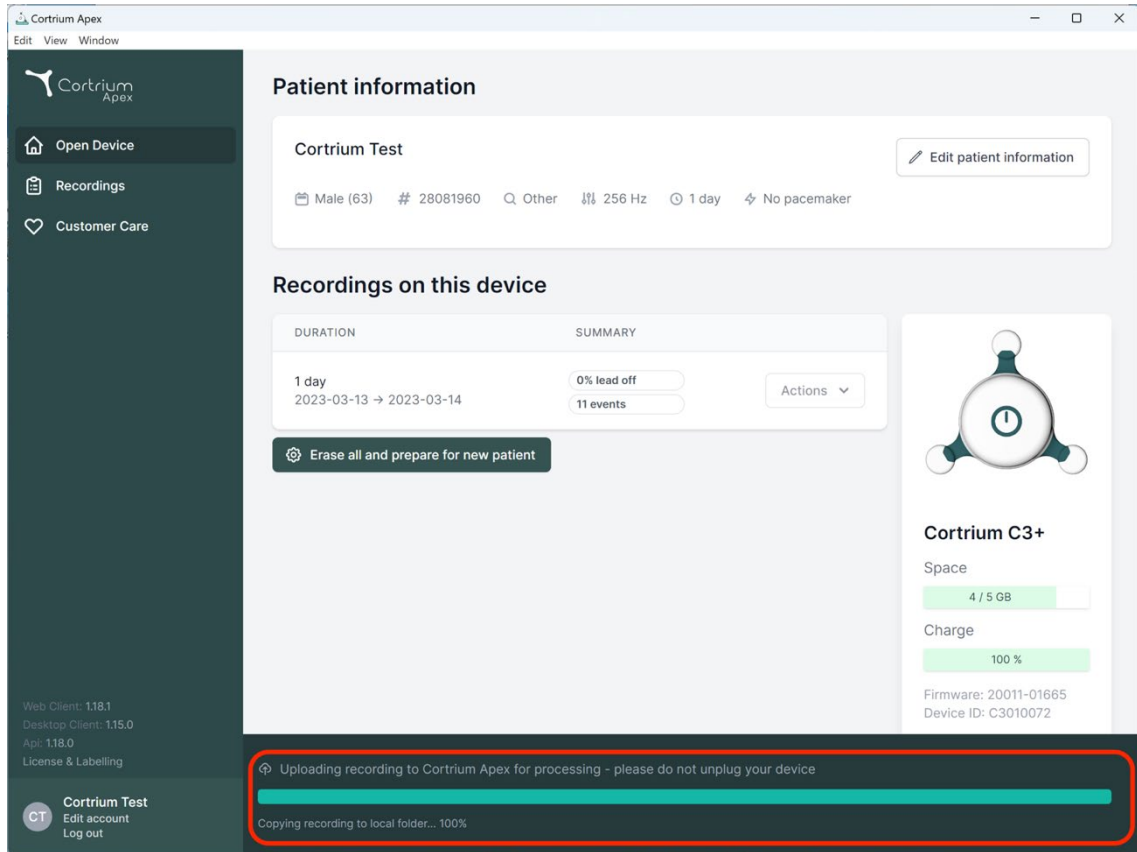
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

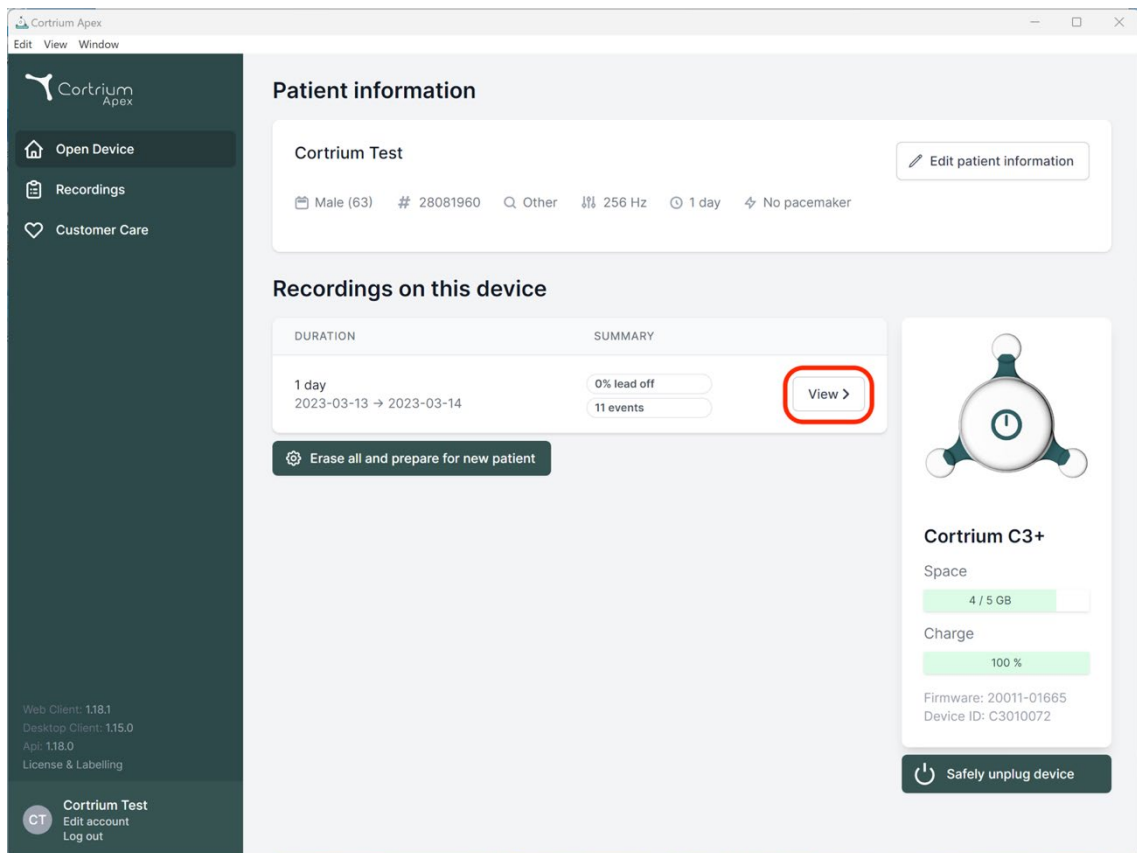
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Web Client: 118.1
Desktop Client: 115.0
Api: 118.0
License & Labelling

Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
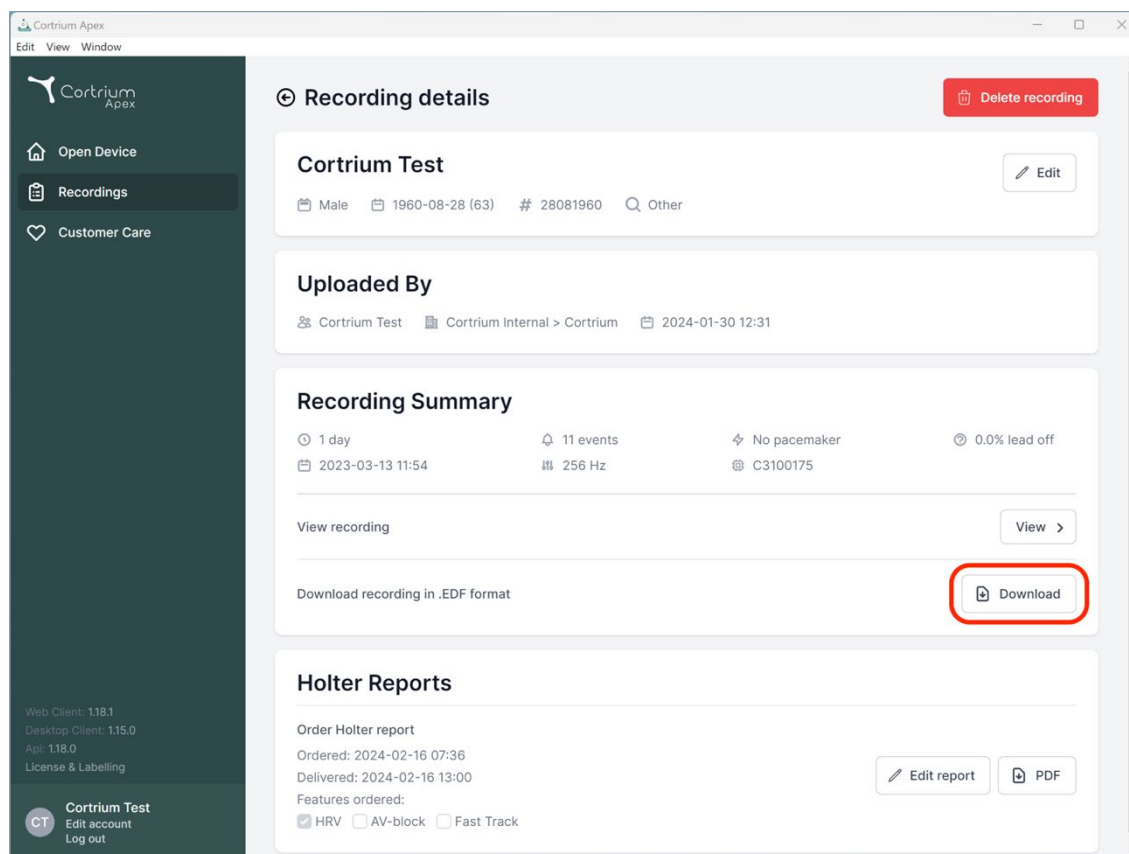
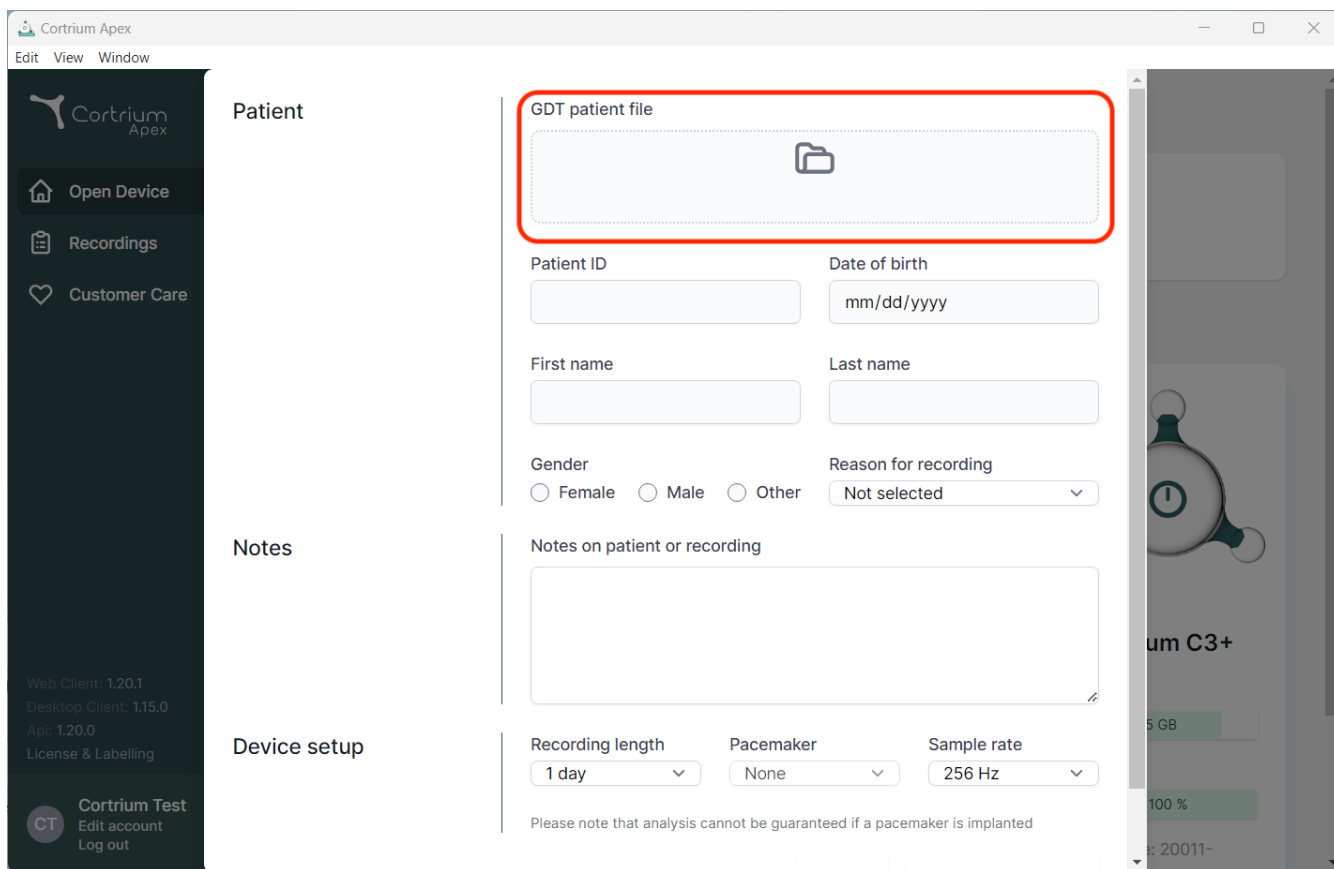
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with the Cortrium Apex logo and navigation links: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'App: 1.20.0', and 'License & Labelling'. At the bottom of the sidebar is a 'Cortrium Test' section with 'Edit account' and 'Log out' options.

The main content area is divided into three sections: 'Patient', 'Notes', and 'Device setup'. The 'Patient' section contains a 'GDT patient file' field (highlighted with a red border) with a folder icon, and form fields for 'Patient ID', 'Date of birth' (mm/dd/yyyy), 'First name', 'Last name', 'Gender' (Female, Male, Other), and 'Reason for recording' (Not selected). The 'Notes' section has a 'Notes on patient or recording' text area. The 'Device setup' section includes 'Recording length' (1 day), 'Pacemaker' (None), and 'Sample rate' (256 Hz) dropdowns. A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. On the right, a partial view of a device screen shows 'um C3+' and a battery level of '100 %'.

iii. DICOM inkapsulirani izveštaj

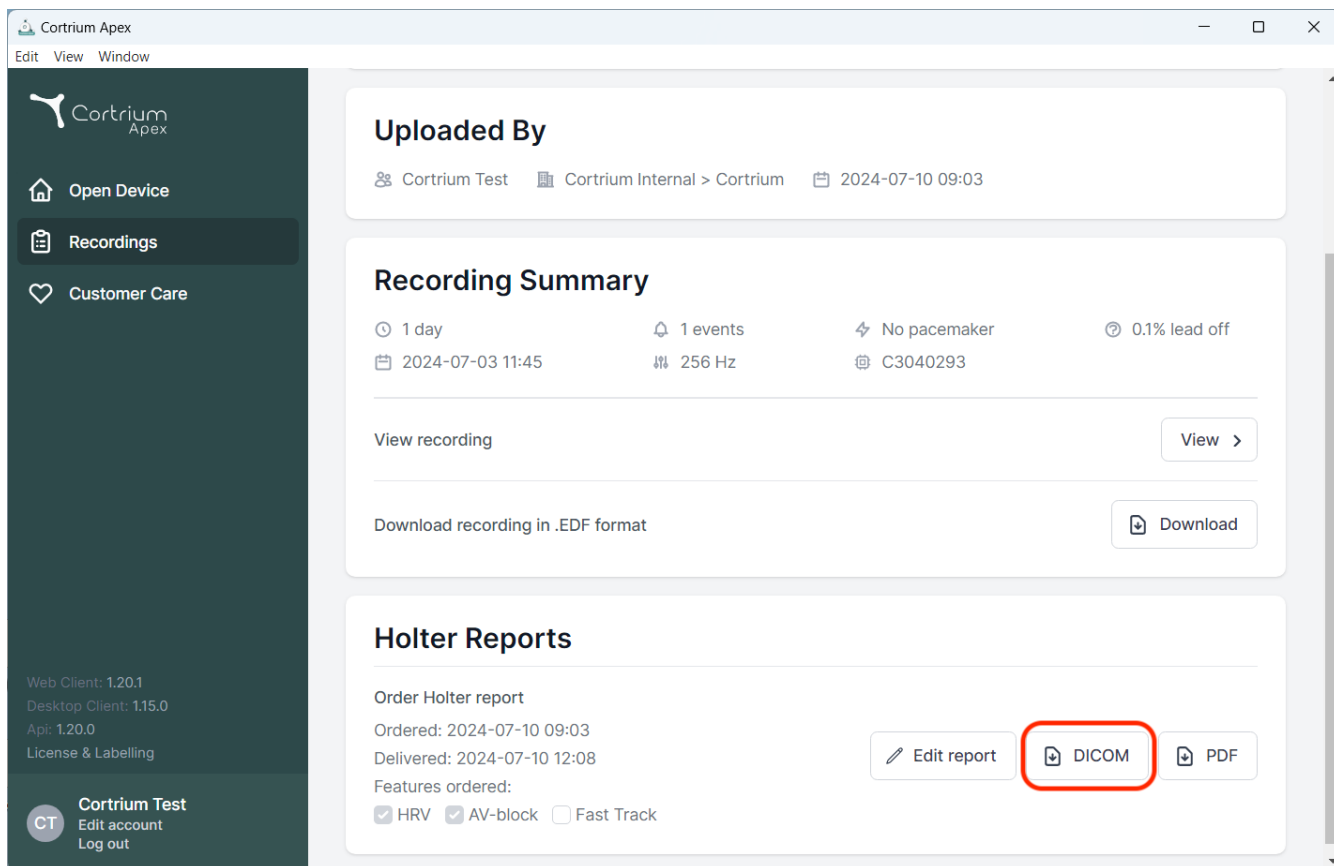- Da biste preuzeli DICOM inkapsulirani izveštaj, kliknite na dugme za preuzimanje u odeljku „Holter izveštaj“ ispod izabranog pacijenta.
- Kada se klikne na dugme, datoteka se preuzima, a naziv datoteke se završava sa .DCM

The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By', 'Recording Summary', and 'Holter Reports'.

Uploaded By: Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.

Recording Summary: Displays recording details: '1 day', '2024-07-03 11:45', '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download' (labeled 'Download recording in .EDF format').

Holter Reports: Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. At the bottom right, there are three buttons: 'Edit report', 'DICOM' (highlighted with a red rectangle), and 'PDF'.

At the bottom left of the sidebar, version information is listed: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', 'Api: 1.20.0', and 'License & Labelling'. A user profile for 'Cortrium Test' is also visible.

- Naziv stanice se podrazumevano podešava na naziv mašine/računara. Ovo se može podesiti na individualnom korisničkom profilu u odeljku „Uređivanje naloga“.

Cortrium Apex

Edit View Window

Open Device

Recordings

Customer Care

Web Client: 1.23.0

Desktop Client: 1.23.0

Api: 1.23.0

License & Labelling

CT

Cortrium Test

Edit account

Log out

Account information

Account information

If you change your email address you need to confirm the new address

Title

Initials

First name

Last name

Email address

Reset password

Account preferences

Here you change preferences for your account

Set preferences

☐

Notify when a Holter report is ready

You will receive a notification by email when a Holter report is ready.

DICOM report preferences

Here you can change preferences for the DICOM report integration

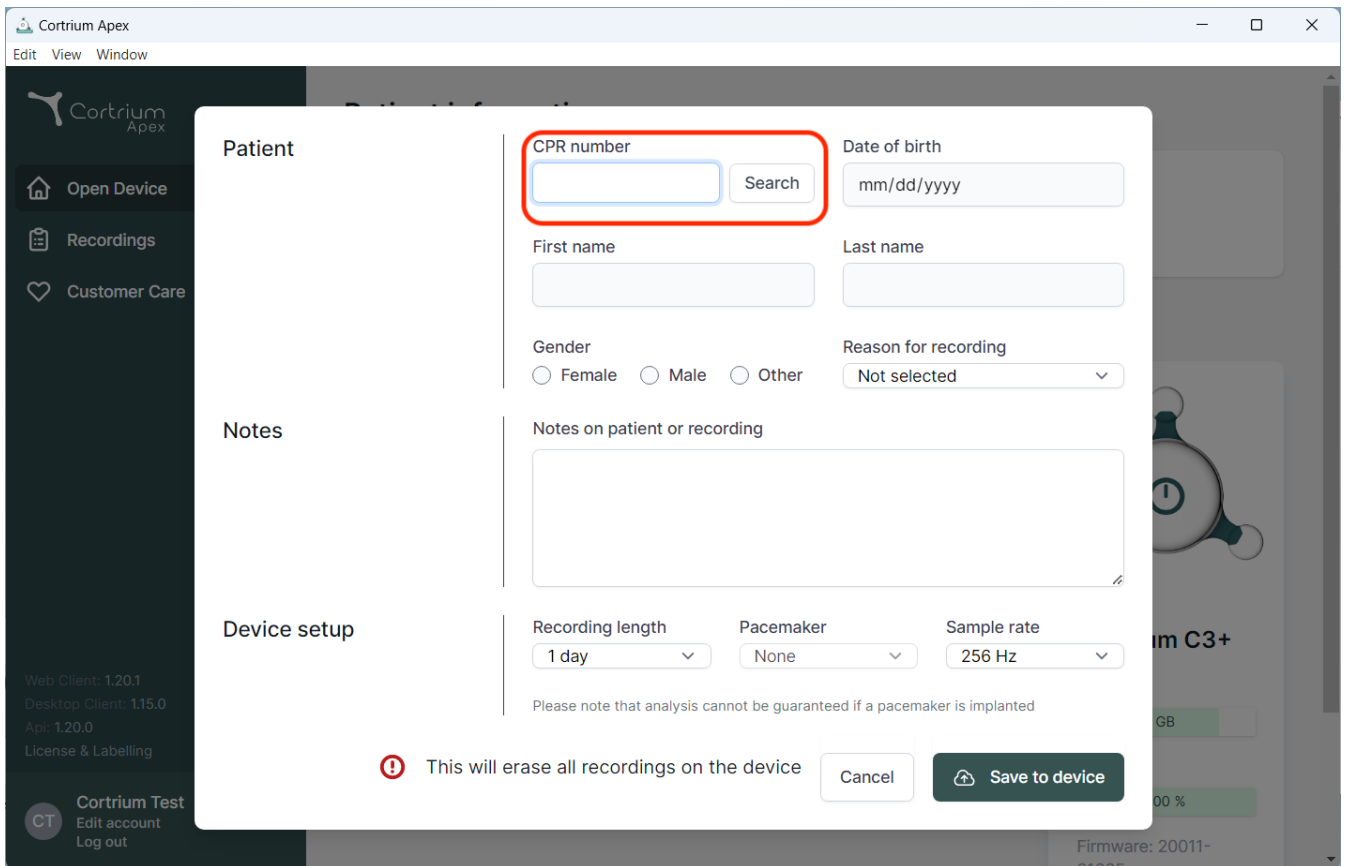
Station name

Leave blank to use computer name

Save changes

iv. Traženje pacijenta po CPR-u

- Unesite danski CPR pacijenta u polje i pritisnite „Pretraži“.
- Informacije o pacijentu se preuzimaju iz registra CPR-a.
- Pribavljene informacije se unose u polja za informacije o pacijentu.
- Učitane informacije se ne mogu uređivati.


The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a form for searching and entering patient information. The form includes the following fields and controls:

- CPR number:** A text input field with a red border, followed by a "Search" button.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.
- Notes:** A section with the label "Notes" and a large text area for "Notes on patient or recording".
- Device setup:** A section with three dropdown menus: "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup section, there is a warning message: "⚠ This will erase all recordings on the device". At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

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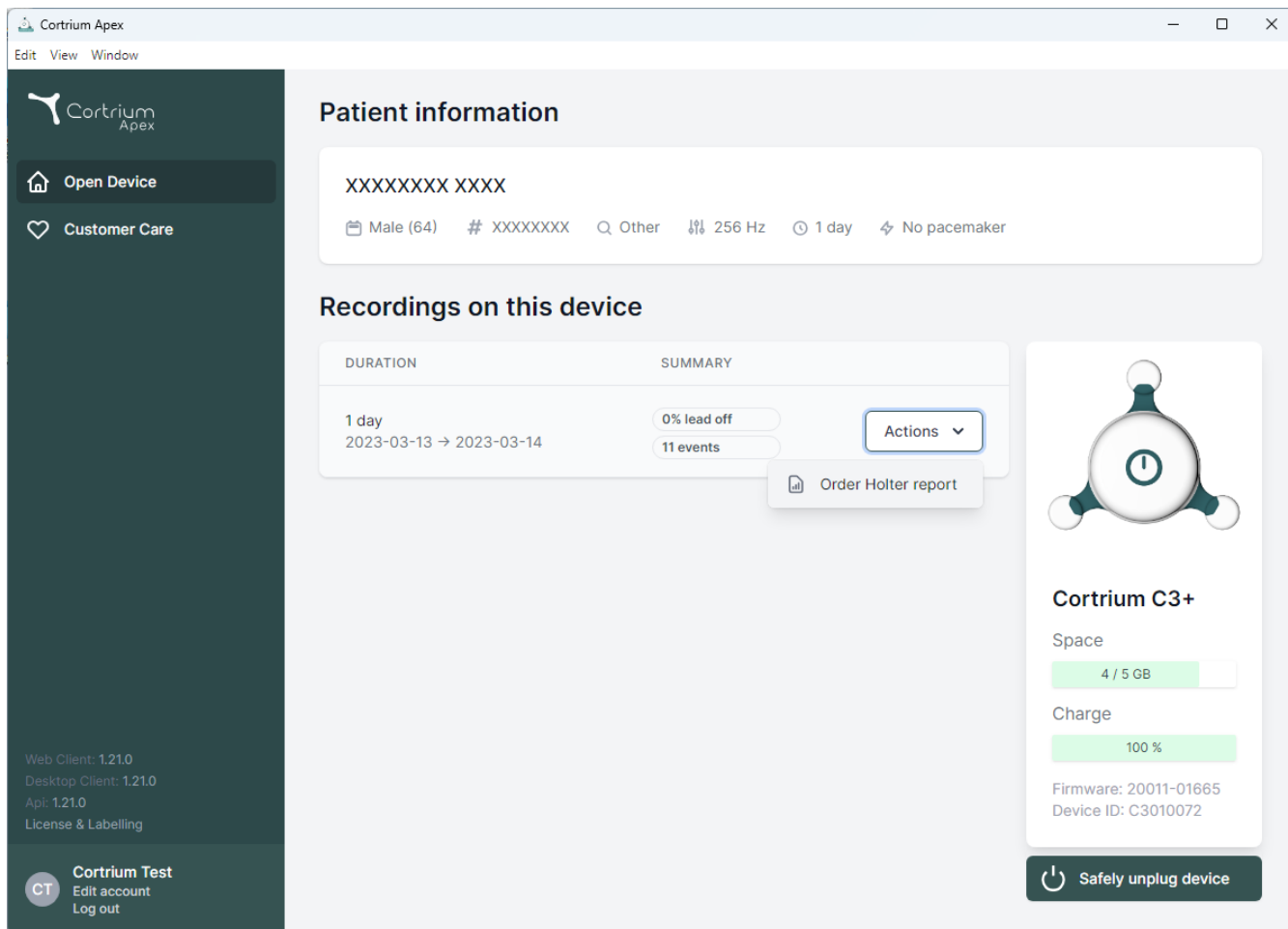The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar features the Cortrium Apex logo, 'Open Device', and 'Customer Care' links. The main content area is divided into two sections: 'Patient information' and 'Recordings on this device'.

Patient information

XXXXXXXX XXXX

Male (64) # XXXXXXXX Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

Actions

Order Holter report

Cortrium C3+

Space
4 / 5 GB

Charge
100 %

Firmware: 20011-01665
Device ID: C3010072

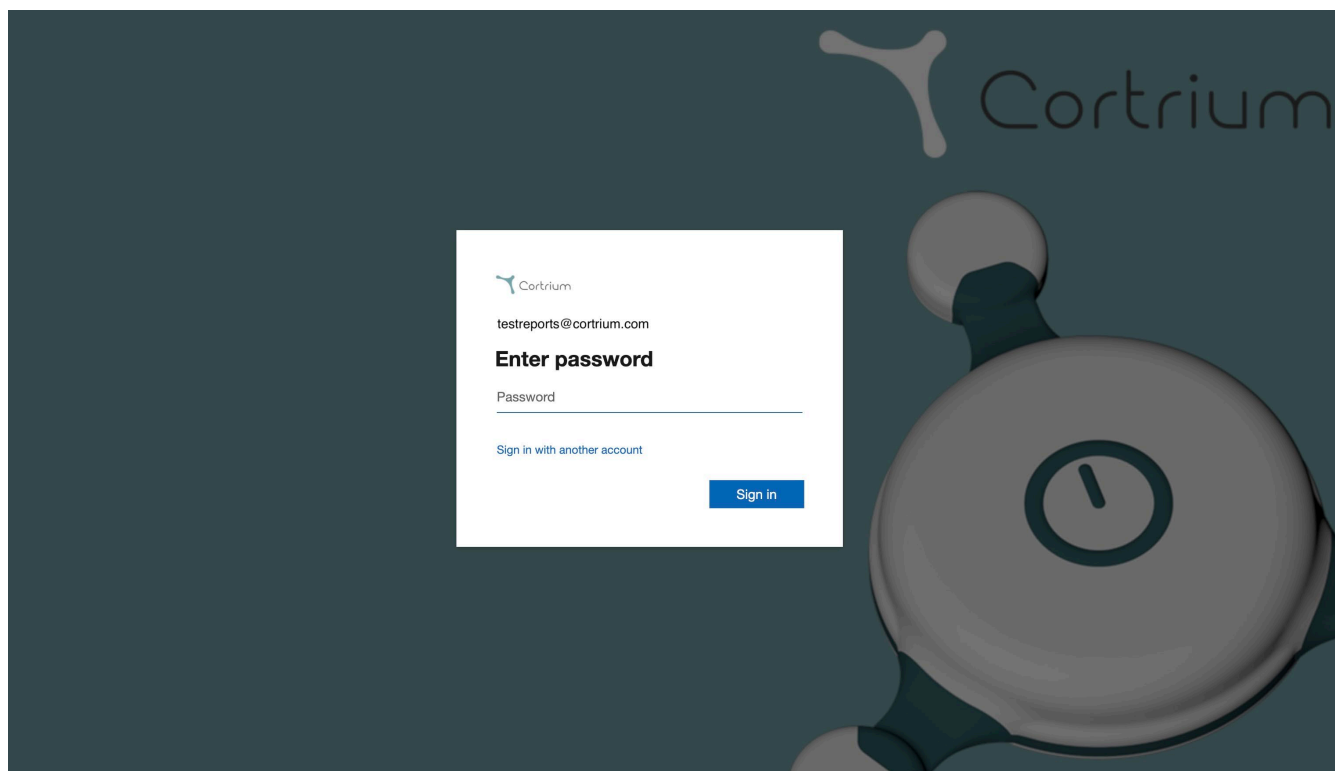
Safely unplug device

Web Client: 1.21.0
Desktop Client: 1.21.0
Api: 1.21.0
License & Labelling

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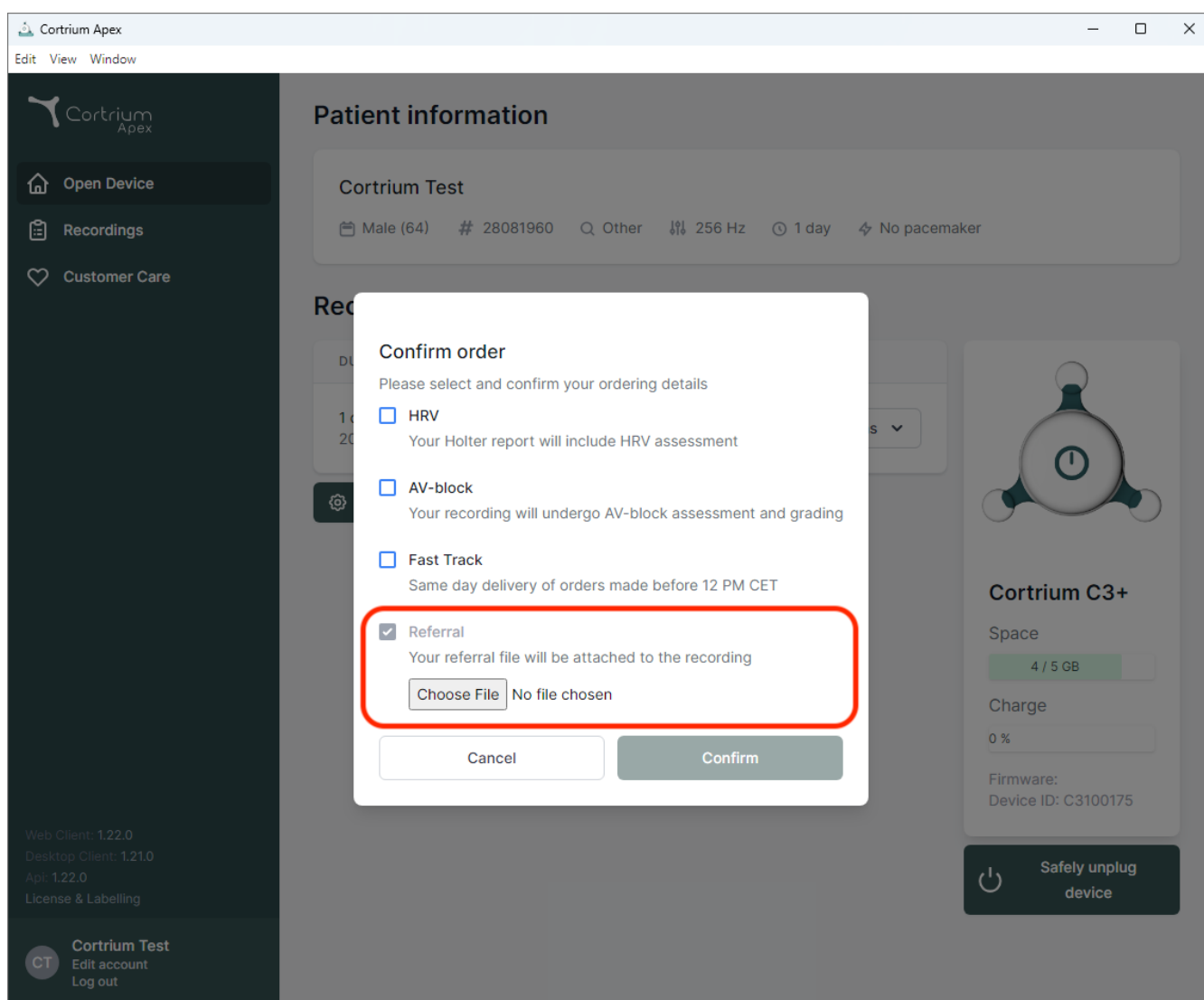
Uploaded By

Test User 03

Distributor 01 > Approver Organisation > Test Clinic

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
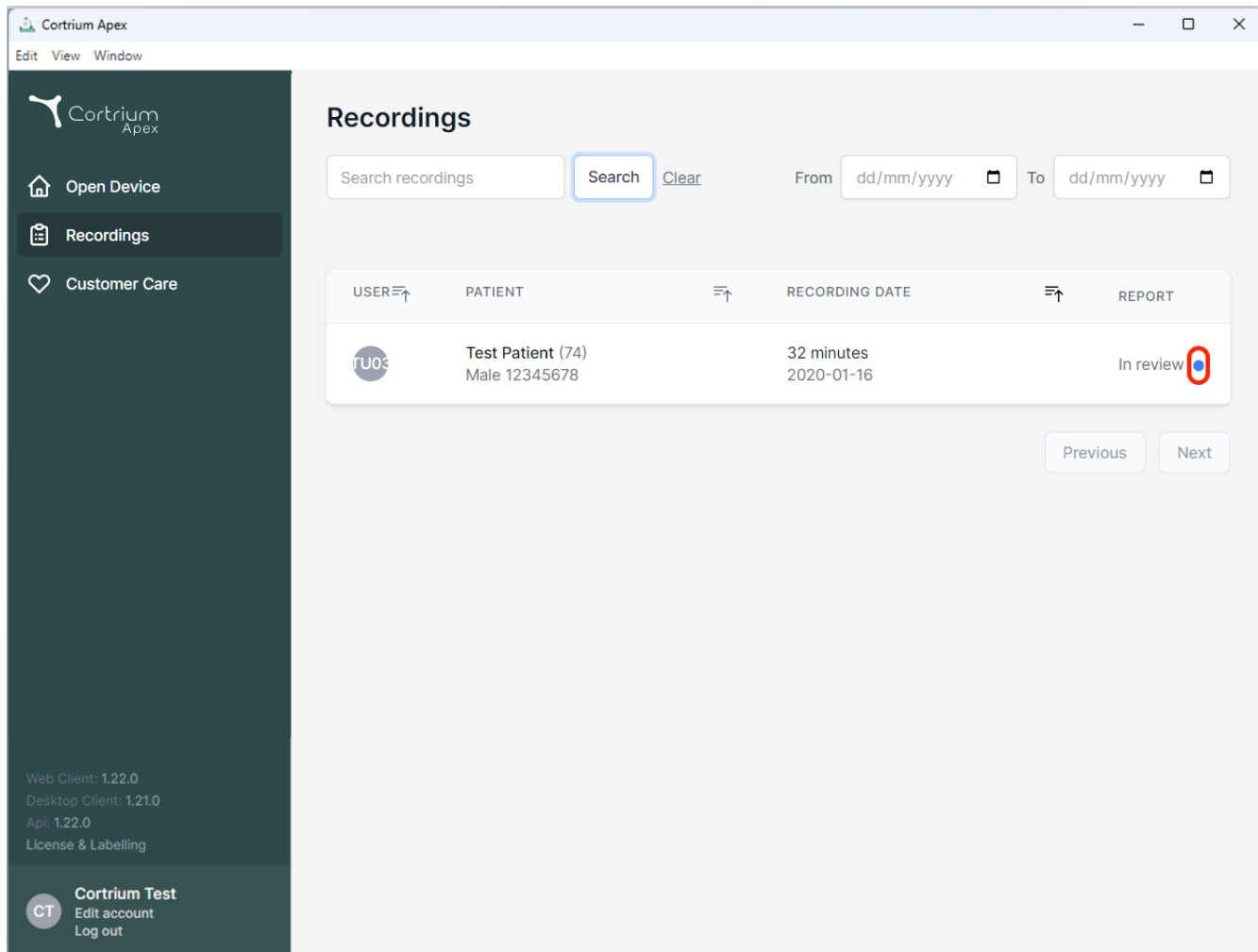
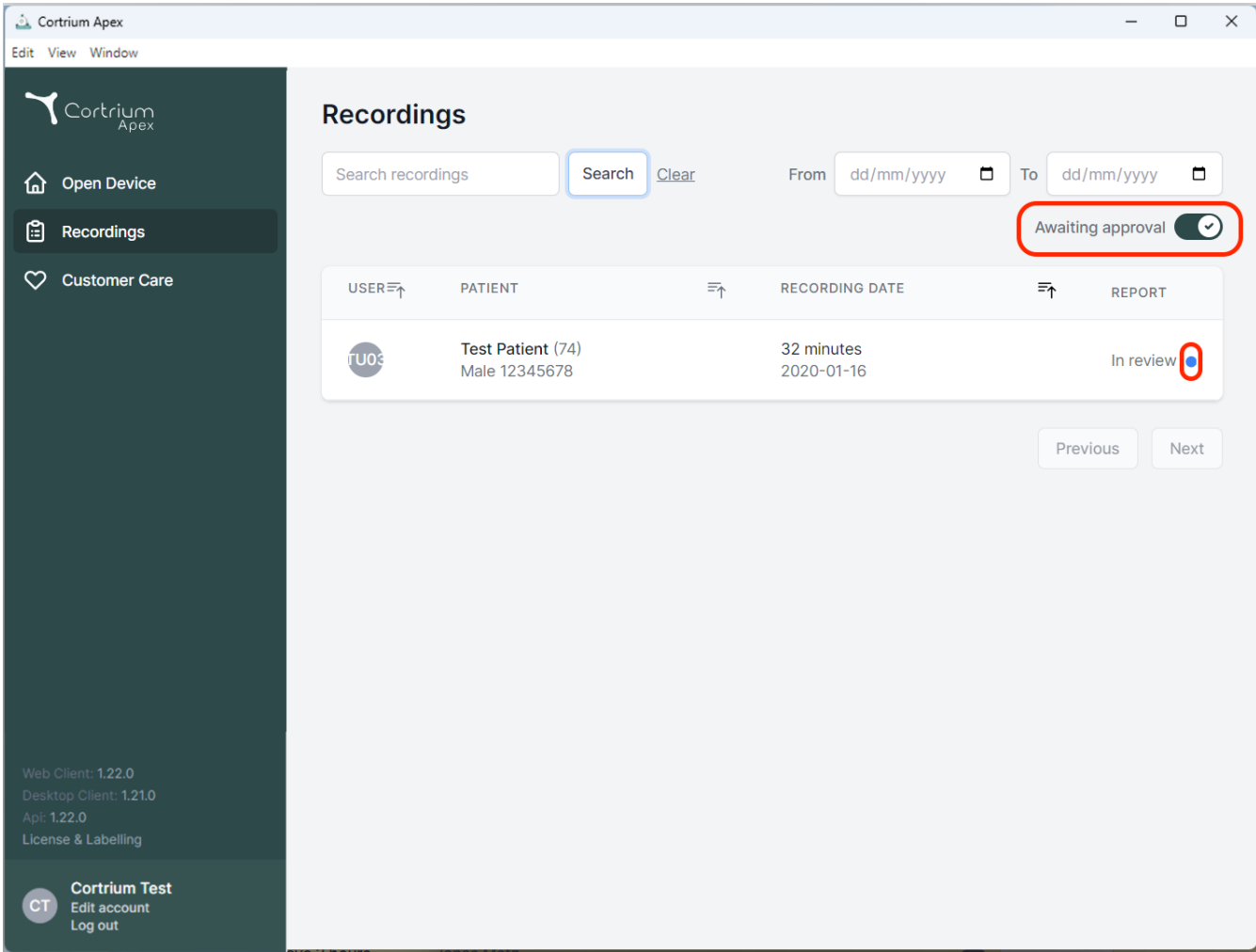
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings (selected), and Customer Care. The main area is titled 'Recordings' and contains a search bar, a 'Search' button, and a 'Clear' link. Below the search bar is a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains one entry for 'Test Patient (74)' with a recording date of '2020-01-16' and a status of 'In review'. A red circle highlights the 'In review' status. In the top right corner of the main area, there is a toggle switch labeled 'Awaiting approval' which is currently turned on, also highlighted with a red circle. At the bottom of the sidebar, there is a 'Cortrium Test' section with links for 'Edit account' and 'Log out'.

USER	PATIENT	RECORDING DATE	REPORT
TU03	Test Patient (74) Male 12345678	32 minutes 2020-01-16	In review

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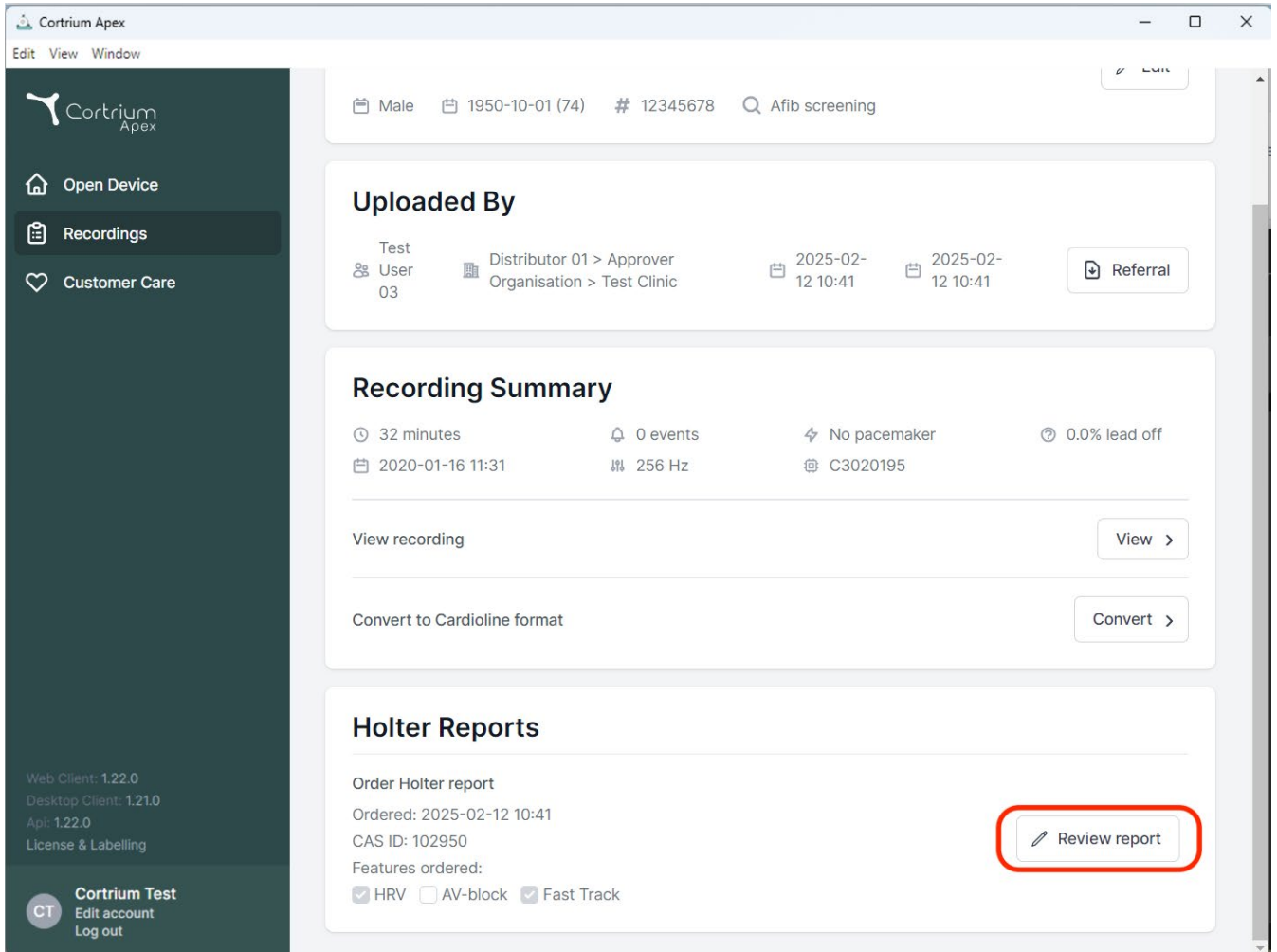
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window' options. The left sidebar contains navigation links: 'Open Device', 'Recordings' (highlighted), and 'Customer Care'. The main content area is divided into several sections:

- Header Information:** Patient details including 'Male', '1950-10-01 (74)', '# 12345678', and 'Afib screening'.
- Uploaded By:** A section showing the test user (User 03), distributor (Distributor 01 > Approver Organisation > Test Clinic), and upload dates (2025-02-12 10:41). A 'Referral' button is present.
- Recording Summary:** A section providing key recording metrics: 32 minutes, 0 events, No pacemaker, and 0.0% lead off. It also includes the recording date (2020-01-16 11:31), frequency (256 Hz), and ID (C3020195). Below this are buttons for 'View recording' and 'Convert to Cardioline format'.
- Holter Reports:** A section for managing holter reports. It shows the 'Order Holter report' status, the order date (2025-02-12 10:41), and the CAS ID (102950). Under 'Features ordered', there are checkboxes for 'HRV' (checked), 'AV-block' (unchecked), and 'Fast Track' (checked). A 'Review report' button is highlighted with a red rectangle.

The bottom of the sidebar contains version information (Web Client: 1.22.0, Desktop Client: 1.21.0, Api: 1.22.0, License & Labelling) and a 'Cortrium Test' section with links for 'Edit account' and 'Log out'.

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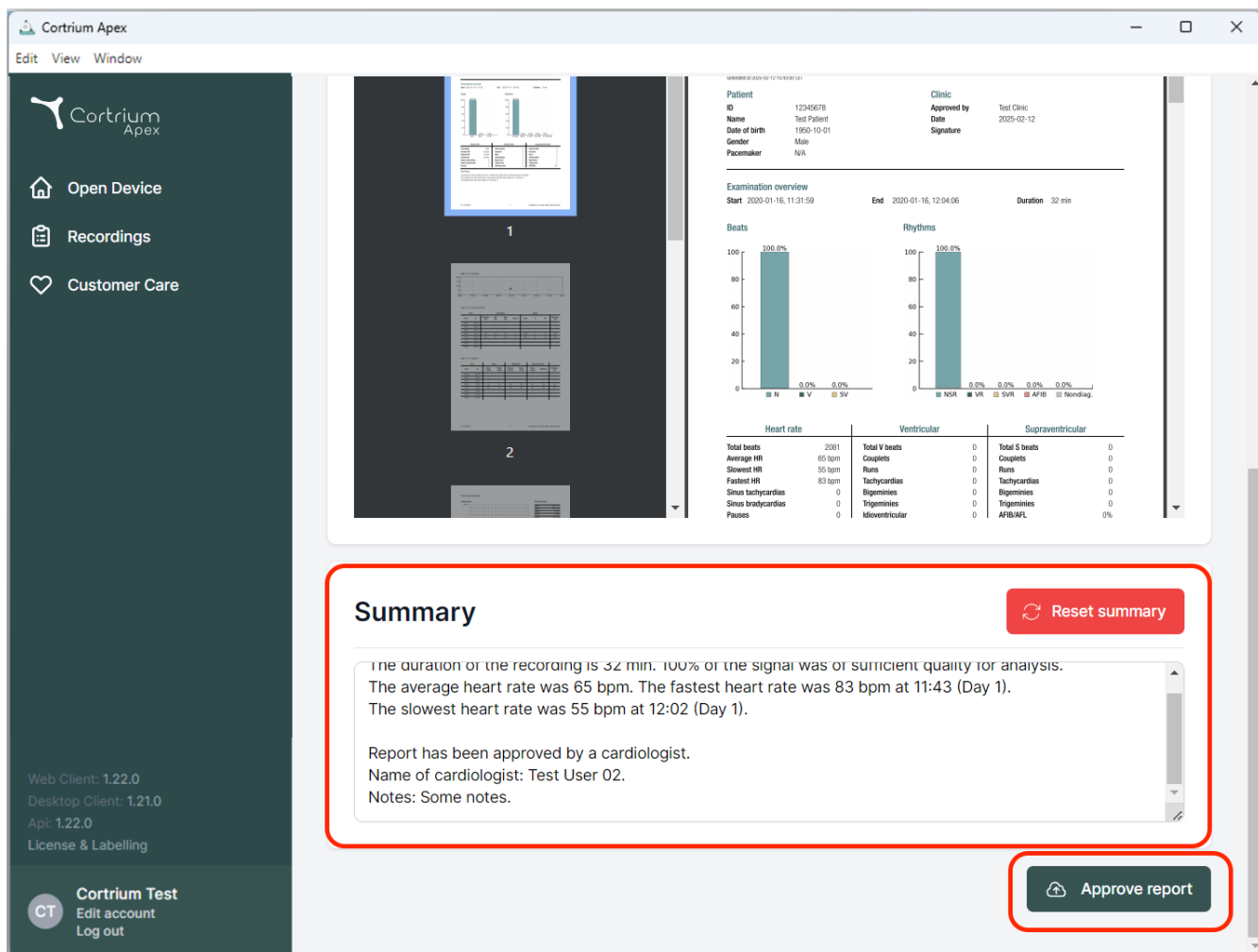The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation links: "Open Device", "Recordings", and "Customer Care". The main content area shows a patient's report for "Test Patient" (ID: 12345678, DOB: 1950-10-01, Male, Pacemaker). The report includes an "Examination overview" with start/end times and duration (32 min). Below this are two bar charts: "Beats" (showing 100.0% for N, 0.0% for V, 0.0% for SV) and "Rhythms" (showing 100.0% for NSR, 0.0% for VR, 0.0% for SVR, 0.0% for AFB, and 0.0% for Noirdiag). A table summarizes heart rate and rhythm data. At the bottom, a "Summary" box (highlighted with a red border) contains the following text:

Summary Reset summary

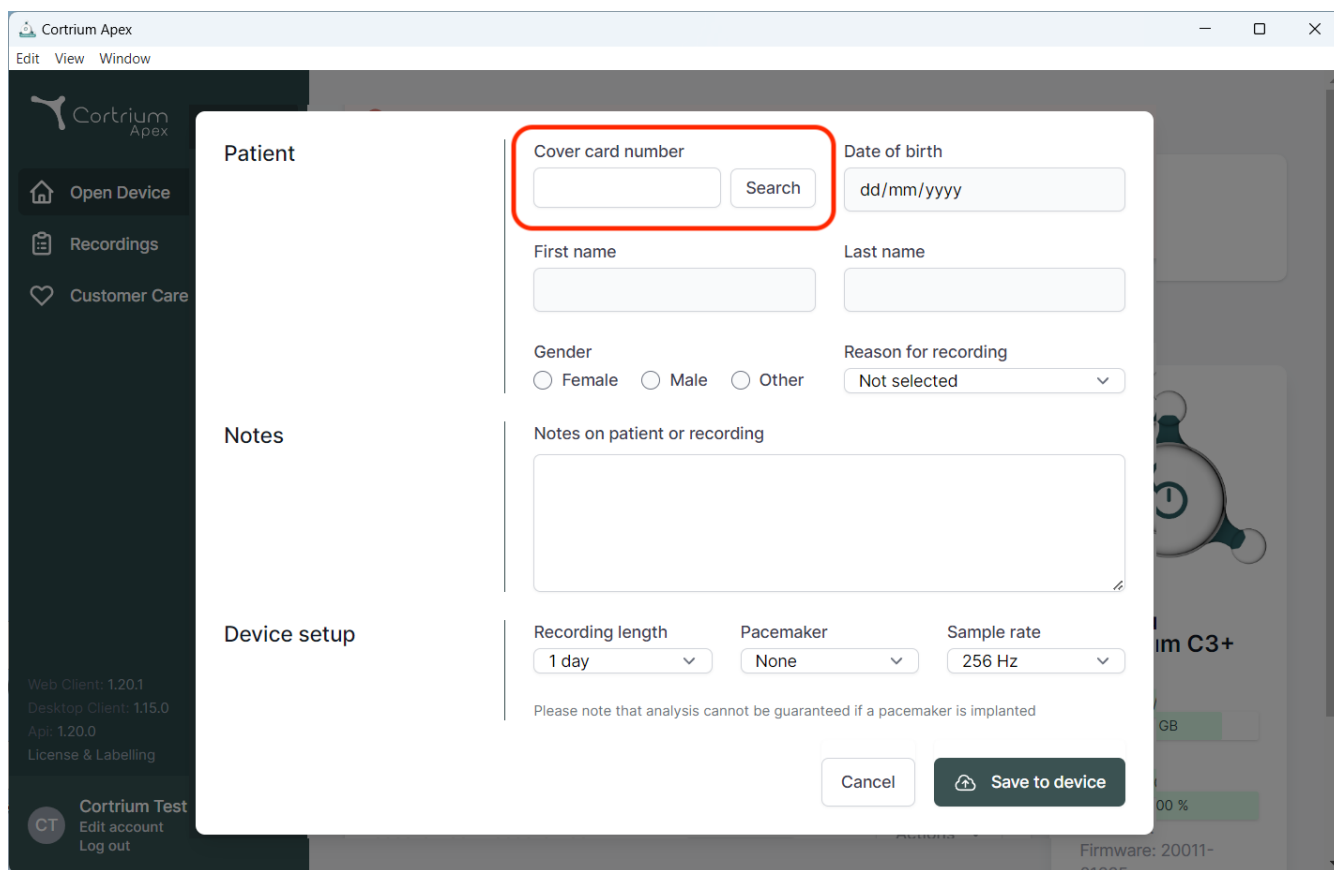
The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
 The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
 The slowest heart rate was 55 bpm at 12:02 (Day 1).

Report has been approved by a cardiologist.
 Name of cardiologist: Test User 02.
 Notes: Some notes.

At the bottom right, there is a button labeled "Approve report" (highlighted with a red border).

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The screenshot displays the Cortrium Apex software interface. A modal window titled "Patient" is open, containing the following fields and controls:

- Cover card number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangular box.
- Date of birth:** A text input field with a placeholder "dd/mm/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes on patient or recording:** A large text area for notes.
- Device setup:**
 - Recording length:** A dropdown menu showing "1 day".
 - Pacemaker:** A dropdown menu showing "None".
 - Sample rate:** A dropdown menu showing "256 Hz".

At the bottom of the modal, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". Below this note are two buttons: "Cancel" and "Save to device".

The background interface shows a sidebar with "Open Device", "Recordings", and "Customer Care" options. The bottom left corner displays version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "Api: 1.20.0", and "License & Labelling". The bottom right corner shows "Cortrium Test", "Edit account", "Log out", and "Firmware: 20011-01000".

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