# Cortrium Apex 1.16

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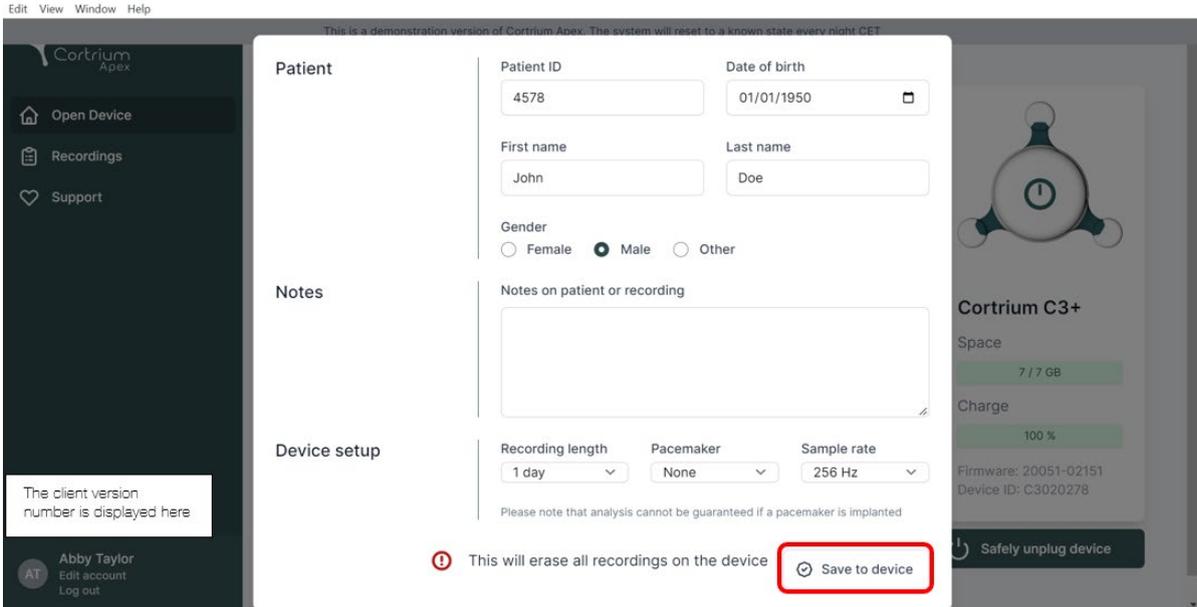
The screenshot displays the Cortrium Apex software interface. On the left is a dark sidebar with navigation options: "Open Device", "Recordings", and "Support". Below these is a user profile for "Abby Taylor" with options to "Edit account" and "Log out". The main area is titled "Patient" and contains several sections: "Patient" information (ID: 4578, Date of birth: 01/01/1950, First name: John, Last name: Doe, Gender: Male), "Notes" (a text area for "Notes on patient or recording"), and "Device setup" (Recording length: 1 day, Pacemaker: None, Sample rate: 256 Hz). A red box highlights the "Save to device" button. A warning message states: "This will erase all recordings on the device". On the right, a "Cortrium C3+" device status panel shows "Space: 7 / 7 GB", "Charge: 100%", and "Firmware: 20051-02151, Device ID: C3020278". A "Safely unplug device" button is also visible.

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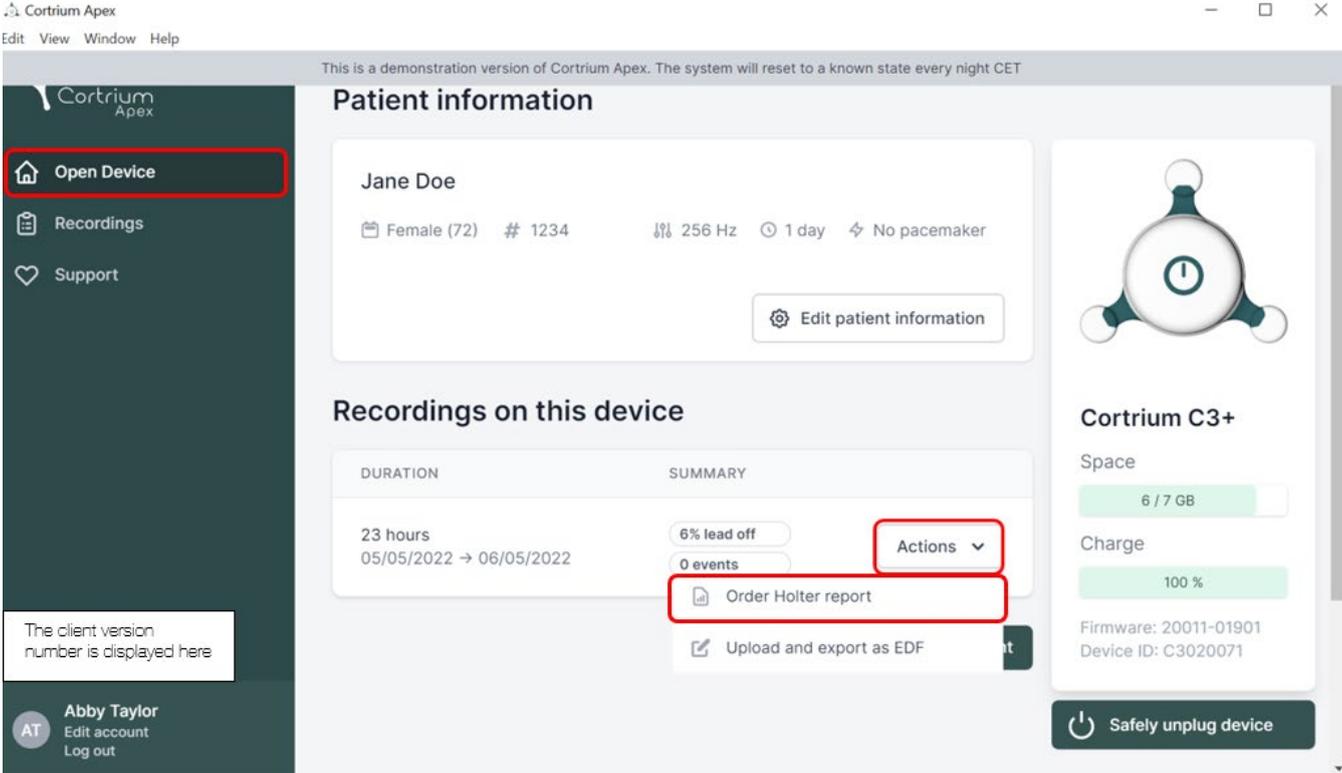
The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit View Window Help' and a status message: 'This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET'. The main content area is divided into three sections:

- Patient information:** Displays 'Jane Doe', 'Female (72)', '# 1234', '256 Hz', '1 day', and 'No pacemaker'. An 'Edit patient information' button is present.
- Recordings on this device:** A table with columns 'DURATION' and 'SUMMARY'. The first row shows '23 hours' and '05/05/2022 → 06/05/2022'. The summary includes '6% lead off', '0 events', and an 'Actions' dropdown menu. The 'Order Holter report' option is highlighted.
- Device status (Cortrium C3+):** Shows 'Space' (6 / 7 GB), 'Charge' (100%), 'Firmware: 20011-01901', and 'Device ID: C3020071'. A 'Safely unplug device' button is at the bottom.

The sidebar on the left contains 'Open Device', 'Recordings', and 'Support' buttons. A user profile for 'Abby Taylor' is visible at the bottom left.

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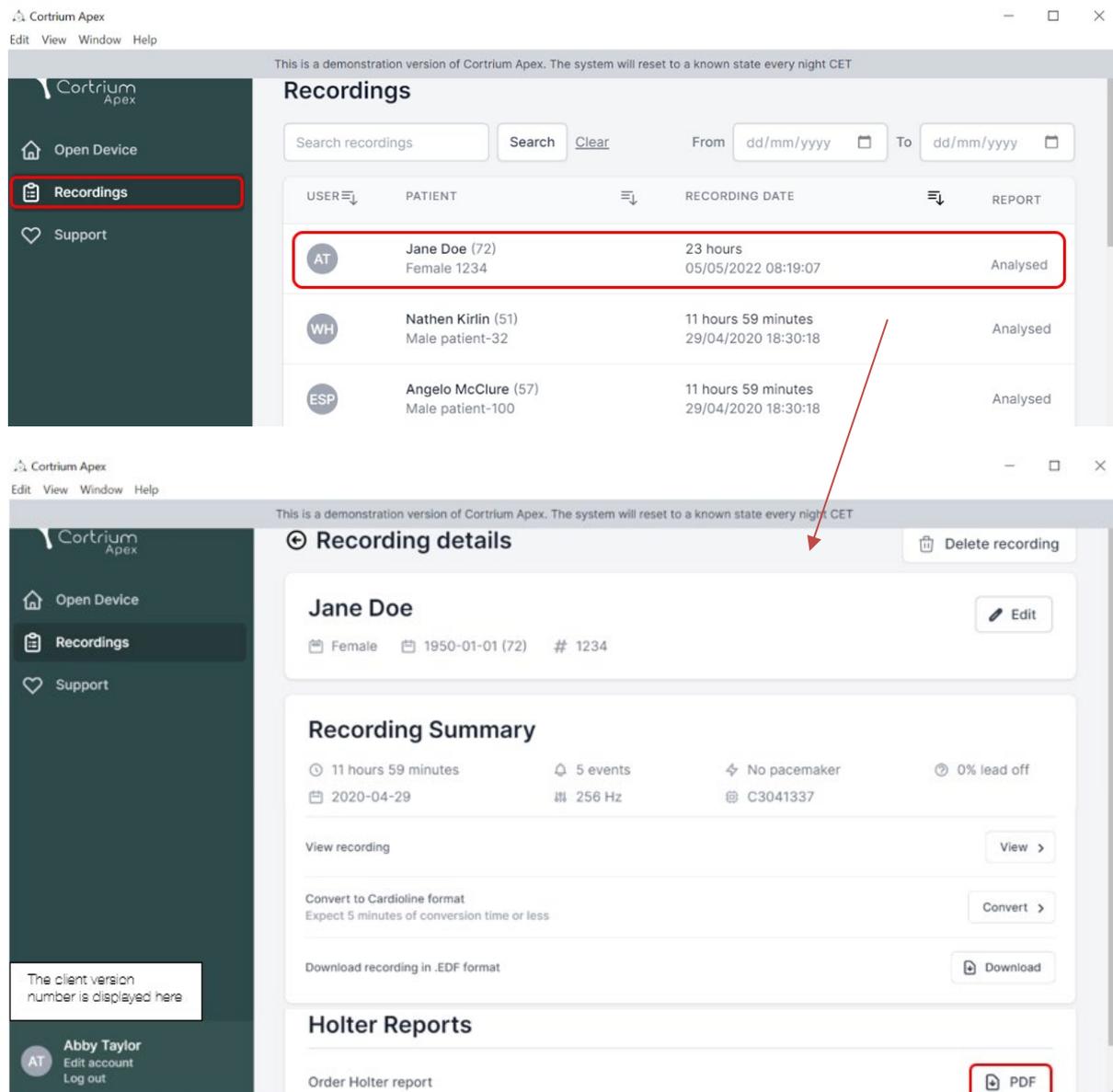
The image shows two screenshots of the Cortrium Apex web application. The top screenshot displays the 'Recordings' page with a table of recording entries. The bottom screenshot shows the 'Recording details' page for a specific recording.

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**Recording details for Jane Doe:**

- Patient:** Jane Doe (72), Female, 1950-01-01, # 1234
- Recording Summary:** 11 hours 59 minutes, 5 events, No pacemaker, 0% lead off, 2020-04-29, 256 Hz, C3041337
- Actions:** View recording, Convert to Cardiolite format, Download recording in .EDF format
- Holter Reports:** Order Holter report (PDF button highlighted)

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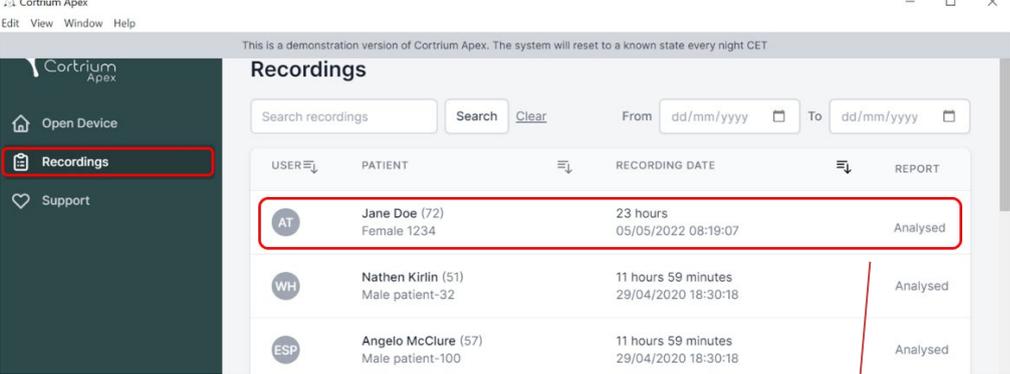
Cortrium Apex  
Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

### Recordings

Search recordings  Search [Clear](#) From  To

USER	PATIENT	RECORDING DATE	REPORT
AT	Jane Doe (72) Female 1234	23 hours 05/05/2022 08:19:07	Analysed
WH	Nathen Kirilin (51) Male patient-32	11 hours 59 minutes 29/04/2020 18:30:18	Analysed
ESP	Angelo McClure (57) Male patient-100	11 hours 59 minutes 29/04/2020 18:30:18	Analysed

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Cortrium Apex  
Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

### Recording details

Delete recording

**Jane Doe** Edit

Female 1950-01-01 (72) # 1234

#### Recording Summary

11 hours 59 minutes 5 events No pacemaker 0% lead off  
2020-04-29 256 Hz C3041337

View recording View >

Convert to Cardiolite format  
Expect 5 minutes of conversion time or less Convert >

Download recording in .EDF format Download

#### Holter Reports

Order Holter report PDF

The client version number is displayed here

Abby Taylor  
AT Edit account Log out

Cortrium Apex  
Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

### Recording details

Delete recording

**Patient**

Patient ID: 1234 Date of birth: 01/01/1950

First name: Jane Last name: Doe

Gender:  Female  Male  Other

**Notes**

Notes on patient or recording

Save

0.9% lead off

View >

Convert >

Download

#### Holter Reports

The client version number is displayed here

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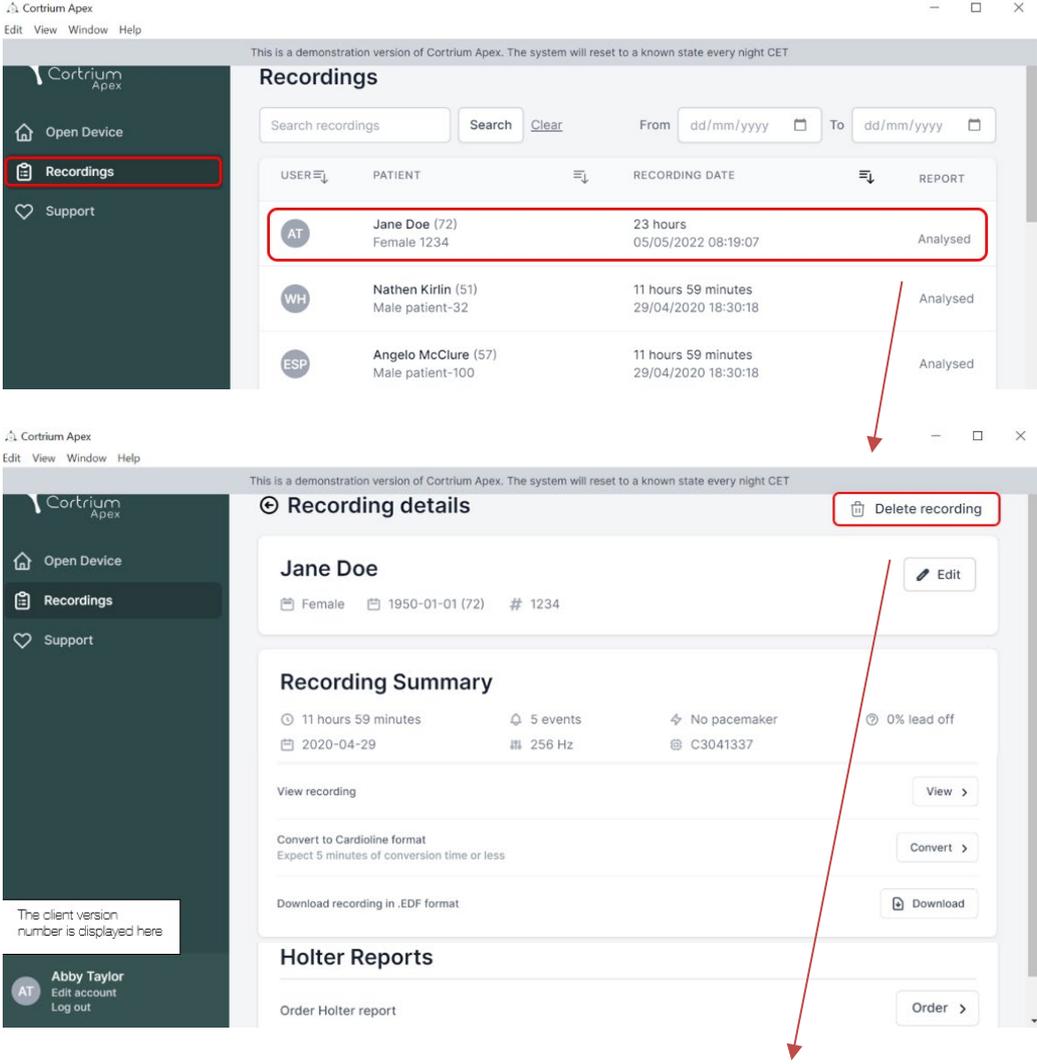
The screenshot displays the Cortrium Apex software interface. The top section shows the 'Recordings' page with a search bar and a table of recordings. The 'Recordings' menu item in the sidebar is highlighted with a red box. The table lists three recordings, with the first one for 'Jane Doe' highlighted. A red arrow points from this row to the 'Recording details' view below. In the 'Recording details' view, the 'Delete recording' button in the top right corner is highlighted with a red box. Another red arrow points from this button to the bottom of the page.

USER	PATIENT	RECORDING DATE	REPORT
AT	Jane Doe (72) Female 1234	23 hours 05/05/2022 08:19:07	Analysed
WH	Nathen Kirlin (51) Male patient-32	11 hours 59 minutes 29/04/2020 18:30:18	Analysed
ESP	Angelo McClure (57) Male patient-100	11 hours 59 minutes 29/04/2020 18:30:18	Analysed

**Recording details**

**Jane Doe**  
Female 1950-01-01 (72) # 1234

**Recording Summary**

11 hours 59 minutes | 5 events | No pacemaker | 0% lead off  
2020-04-29 | 256 Hz | C3041337

View recording [View >](#)

Convert to Cardioline format  
Expect 5 minutes of conversion time or less [Convert >](#)

Download recording in .EDF format [Download](#)

**Holter Reports**

Order Holter report [Order >](#)

Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

### Recording details

**Jane Doe** Edit

Female 1940-01-01 (82) # 1234

**Confirm delete**  
You are removing Jane Doe

To confirm delete, please write 'Jane Doe' in the input field

Cancel Delete

0.9% lead off View >

**Convert to Cardioline format**  
Expect 5 minutes of conversion time or less Convert >

Download recording in .EDF format Download

### Holter Reports

The client version number is displayed here

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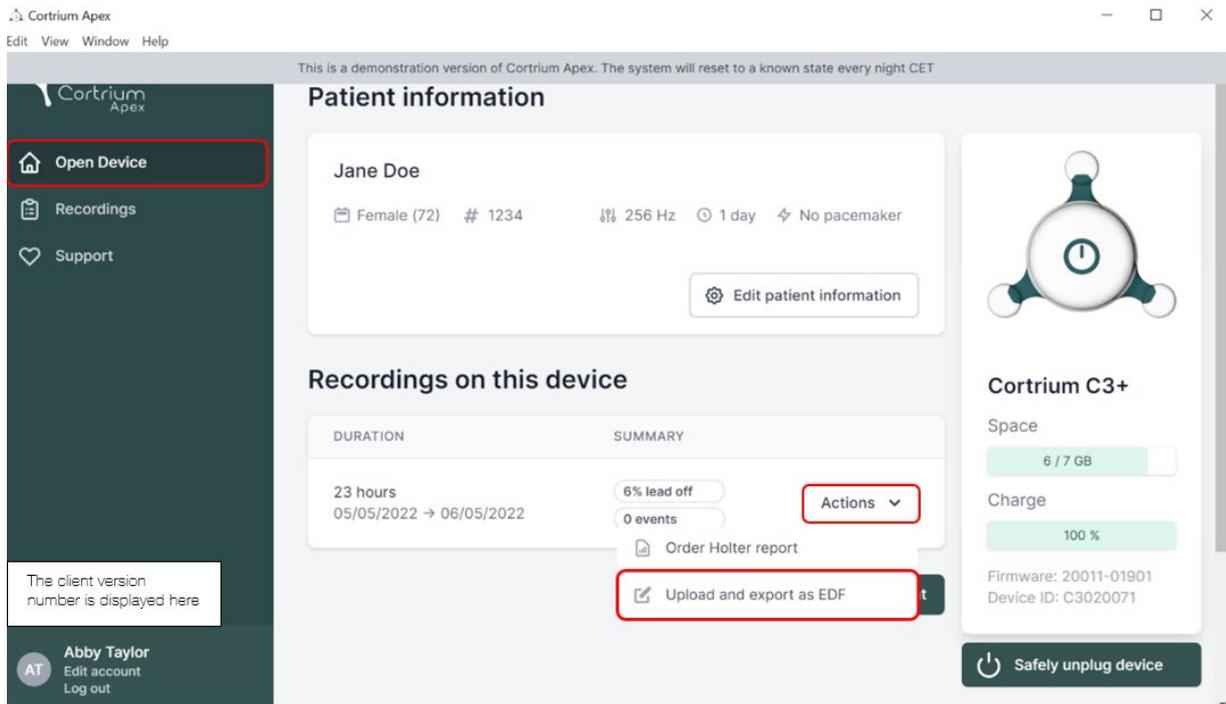
The screenshot shows the Cortrium Apex software interface. The top bar includes the title 'Cortrium Apex' and a menu with 'Edit', 'View', 'Window', and 'Help'. A notification states: 'This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET'. The main content area is divided into three sections:

- Patient information:** Displays 'Jane Doe', 'Female (72)', '# 1234', '256 Hz', '1 day', and 'No pacemaker'. An 'Edit patient information' button is present.
- Recordings on this device:** Shows a recording duration of '23 hours' from '05/05/2022' to '06/05/2022'. The summary indicates '6% lead off' and '0 events'. An 'Actions' dropdown menu is open, with 'Upload and export as EDF' highlighted.
- Device status (Cortrium C3+):** Shows 'Space' at '6 / 7 GB', 'Charge' at '100%', 'Firmware: 20011-01901', and 'Device ID: C3020071'. A 'Safely unplug device' button is at the bottom.

The left sidebar contains 'Open Device', 'Recordings', and 'Support' buttons. A user profile for 'Abby Taylor' is visible at the bottom left.

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Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

Cortrium Apex

Open Device

Recordings

Support

The client version number is displayed here

Abby Taylor  
Edit account  
Log out

### Patient information

Jane Doe

Female (72) # 1234 256 Hz 1 day No pacemaker

Edit patient information

### Recordings on this device

DURATION	SUMMARY
23 hours 05/05/2022 → 06/05/2022	6% lead off 0 events Actions

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 12%

#### Cortrium C3+

Space: 6 / 7 GB

Charge: 100 %

Firmware: 20011-01901  
Device ID: C3020071

Cortrium Apex Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

Cortrium Apex

Open Device

Recordings

Support

The client version number is displayed here

Abby Taylor  
Edit account  
Log out

### Patient information

Jane Doe

Female (72) # 1234 256 Hz 1 day No pacemaker

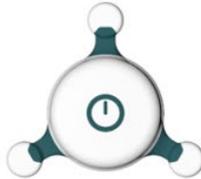
Edit patient information

### Recordings on this device

DURATION	SUMMARY
23 hours 05/05/2022 → 06/05/2022	6% lead off 0 events <b>View</b>

Erase all and prepare for new patient

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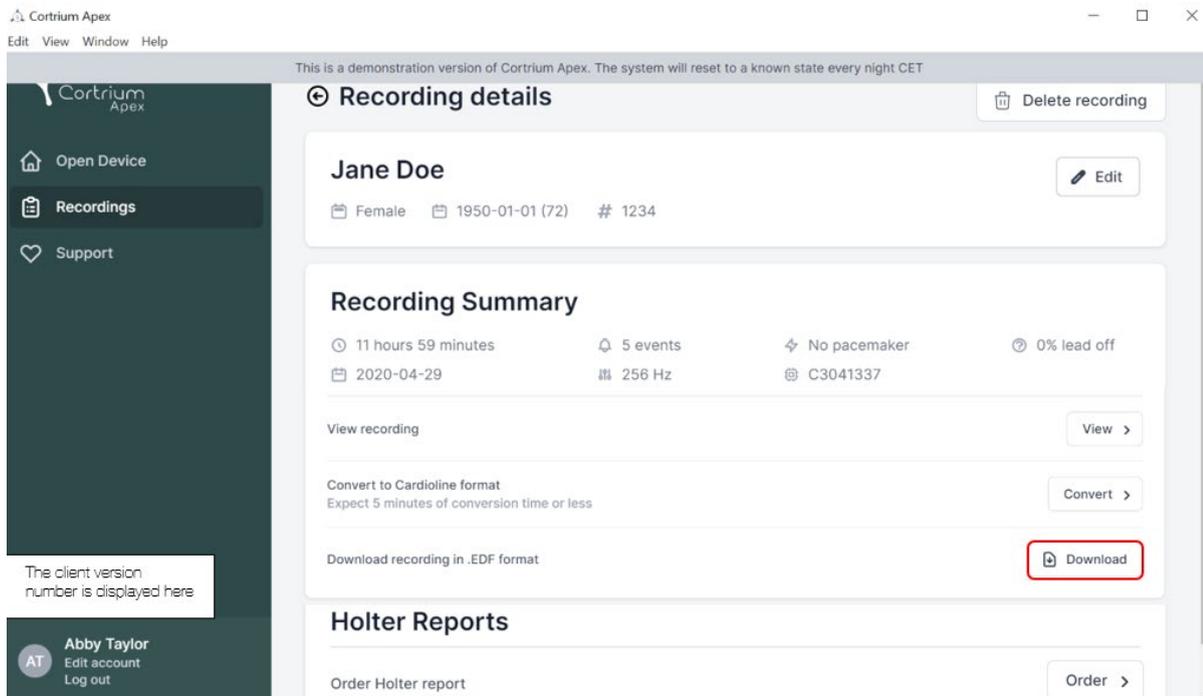
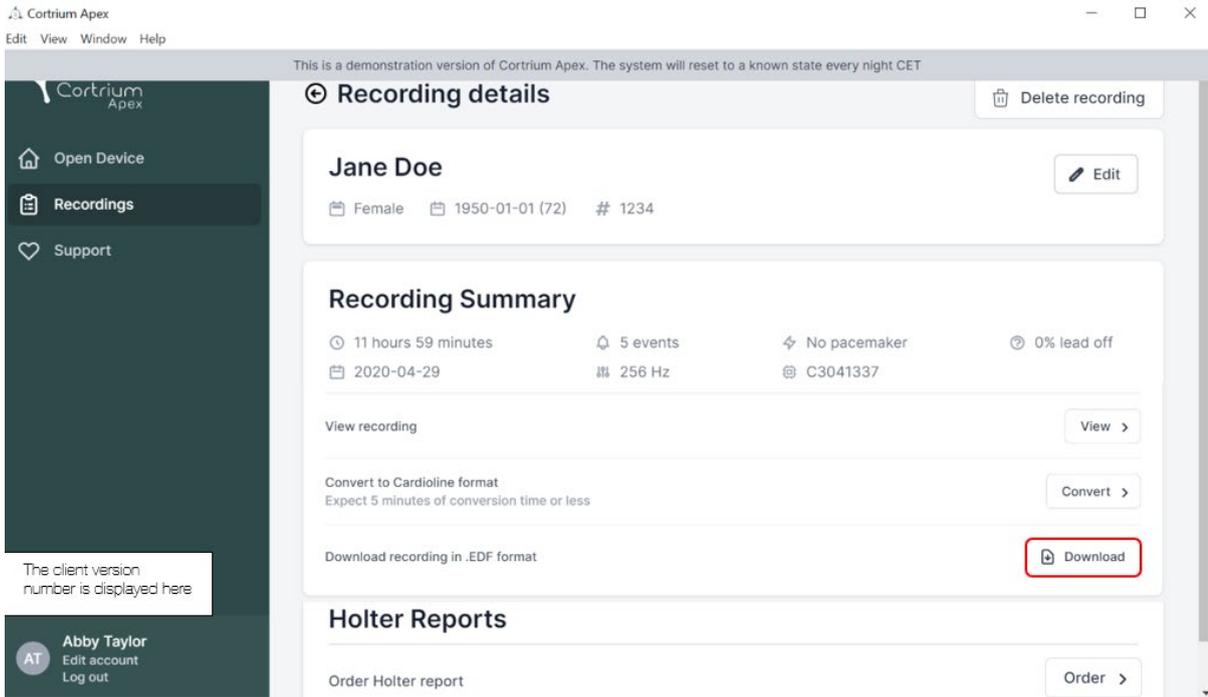
The screenshot shows the Cortrium Apex web application interface. At the top, there is a navigation bar with 'Edit View Window Help' and a status message: 'This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET'. The main content area is titled 'Recording details' and features a 'Delete recording' button. Below this, the patient information for 'Jane Doe' is displayed, including 'Female', '1950-01-01 (72)', and '# 1234', with an 'Edit' button. The 'Recording Summary' section provides key metrics: '11 hours 59 minutes', '5 events', 'No pacemaker', and '0% lead off', along with the date '2020-04-29', '256 Hz', and 'C3041337'. Action buttons include 'View recording', 'Convert to Cardiolite format', and 'Download recording in .EDF format'. The 'Download' button is highlighted with a red box. At the bottom, the 'Holter Reports' section includes an 'Order Holter report' button. A sidebar on the left contains navigation options: 'Open Device', 'Recordings', and 'Support'. A user profile for 'Abby Taylor' is visible at the bottom left, with 'Log out' as an option.

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