

# Cortrium Apex 1.17

## Instructions for Use

1. Intended use and warnings.....	2
2. Installation.....	3
3. Log in to Cortrium Apex.....	4
4. Prepare the C3 <sup>+</sup> for a new patient .....	5
5. Order a Holter report.....	7
6. View Holter reports and the entire recording.....	9
6.1 View Holter reports .....	9
6.2 Recording viewer .....	10
6.3 Summary editor .....	11
7. Edit patient details and view details of recordings.....	11
8. Delete a recording .....	13
9. Download EDF file.....	15
9.1 How to download EDF file before ordering the Holter report .....	15
9.2 How to download EDF file after ordering a report.....	17
10. Log out of Cortrium Apex.....	18
11. Software-Updates .....	18
12. Minimum technical requirements .....	18
13. Troubleshooting .....	19
13.1 "The device seems to be set up for a patient..." .....	19

13.2 Recording summary shows "n/a lead off" or "n/a events" .....	19
13.3 The internet connection was interrupted.....	19
13.4 Apex is newly installed but does not open .....	20
14. Security information .....	20
15. Labelling and symbols.....	21
16. Infrastructure statement .....	21

## 1. Intended use and warnings

### Intended use

Cortrium Apex is a software application used to prepare the Cortrium C3<sup>+</sup> Holter monitor prior to use for a new patient, manage patient data, order, and view ECG Holter reports, download the recording in .EDF format, and to view the entire ECG recording. In addition, recording data can be downloaded and imported into third-party software for analysis. The software is compatible with both Microsoft Windows and MacOS (see section 12 for minimum requirements).

### Intended users

The software is intended to be used by healthcare professionals in private practice, clinics and hospitals as well as remotely.

### Warnings

- This product may only be used in accordance with its intended use as mentioned in these instructions
- Users of this product are unique, and should never share passwords
- Screenshots of the software may contain sensitive information and should be avoided
- Data shared outside of the system are at the user's own risk

## 2. Installation

- a. You have been invited to Cortrium Apex by email. Open the invitation email on a computer where you want to install Cortrium Apex
- b. Click on the link in the invitation email to set a password
- c. Confirm your password
- d. Select "Go to next step" to proceed to the download
- e. Download the installation file for Windows or MacOS
- f. When the download is complete, run the installation file
- g. Once the installation is complete, Cortrium Apex will open automatically
- h. You can now log in using your email address and the respective password and start using Cortrium Apex



Please note that Cortrium Apex requires an internet connection. You will not be able to log in or use Cortrium Apex without an internet connection

### 3. Log in to Cortrium Apex

To use Cortrium Apex, you must first have a user login. If you are the administrator of the facility, this has been set up for you by Cortrium. If you are not the administrator and do not yet have a user login, this can be set up by the administrator of your facility.

Once you have installed Cortrium Apex and received your login, you can proceed with the following steps:


- Open Cortrium Apex on your computer by double-clicking on the Cortrium Apex icon.
- Click on "Sign in" (If you have forgotten your password, click on "Forgot your password")
- Enter your user information and press "Continue".

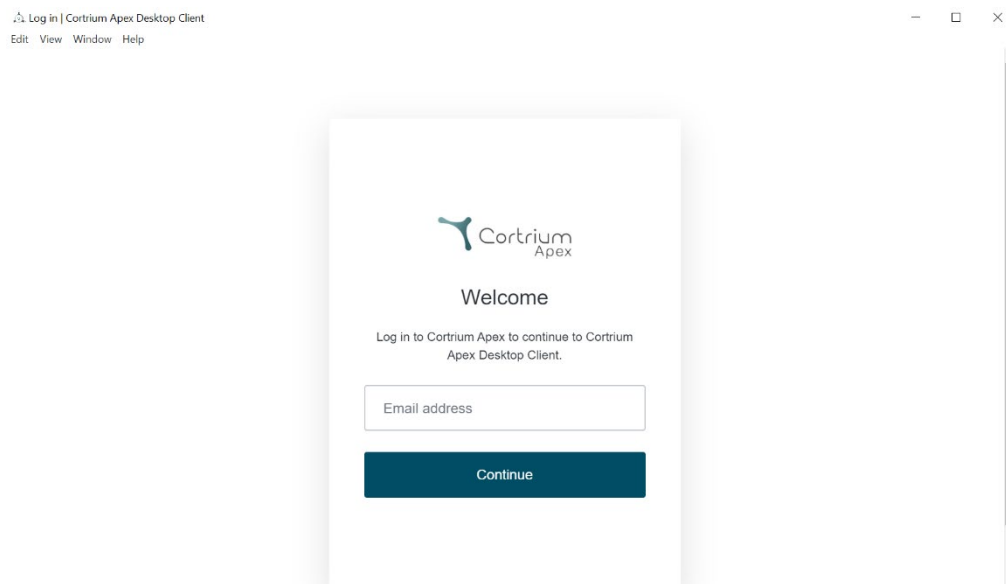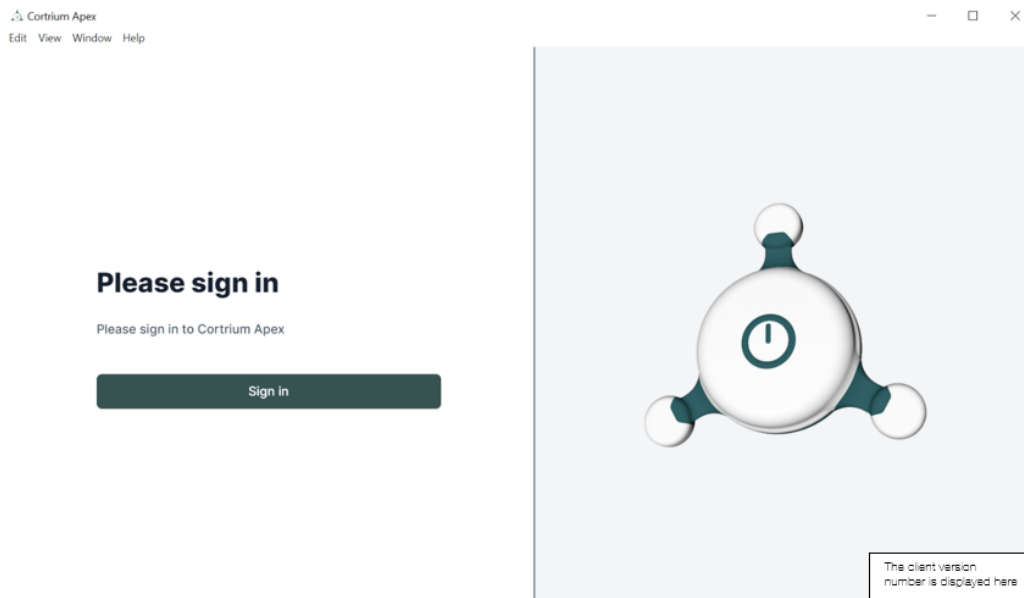


## 4. Prepare the C3<sup>+</sup> for a new patient

If the C3<sup>+</sup> has already been used to record a long-term ECG, please read the section "Order a Holter report" before following the steps in this section.

- Select the "Open Device" button.

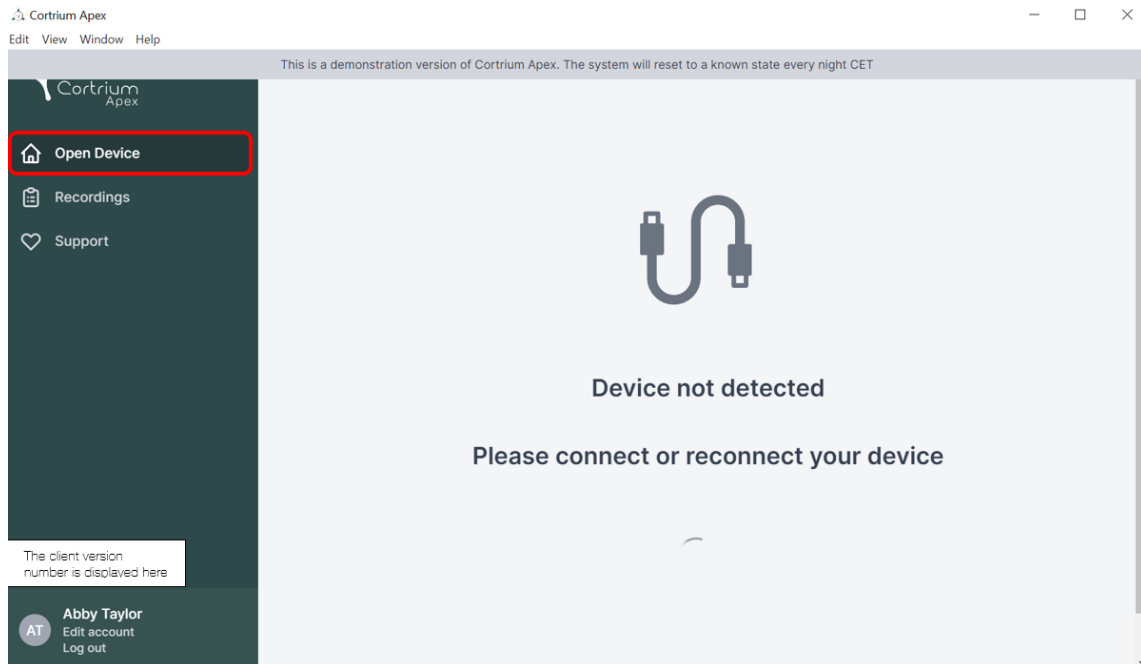


- Connect the C3<sup>+</sup> to the computer using the USB cable provided.  
(If you are using an Apple product, you may need a USB converter)
- Cortrium Apex displays information about the connected device and any data stored on it.

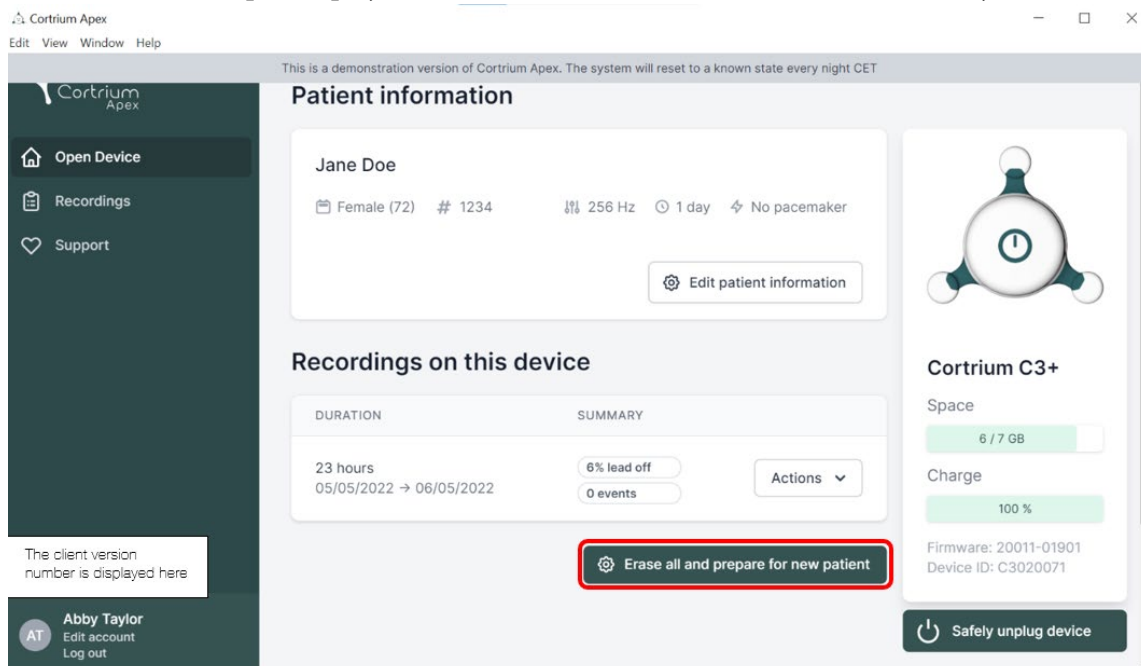

- Select "Erase all and prepare device for new patient".
- Fill in the relevant patient and recording information:

- Now click on "Save to Device". This deletes all patient data or recordings stored on the device and saves the new patient information in the system.

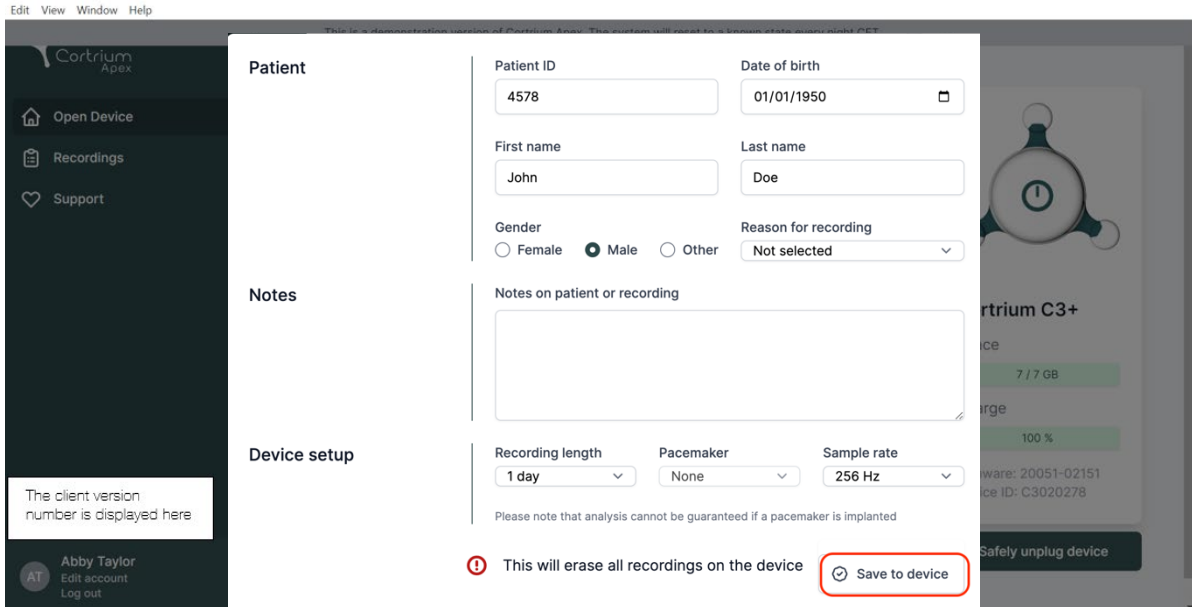

Cortrium Apex

Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will revert to a known state upon reboot.

**Patient**

Patient ID: 4578 Date of birth: 01/01/1950

First name: John Last name: Doe

Gender: ☐ Female ☒ Male ☐ Other Reason for recording: Not selected

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

**Warning:** This will erase all recordings on the device

**Save to device**

The client version number is displayed here

Abby Taylor  
Edit account  
Log out

Cortrium C3+

7 / 7 GB

100 %

Software: 20051-02151  
Device ID: C3020278


Safely unplug device

- You can now eject your device via the "Safely unplug device" button on the right-hand side.
- The C3+ is now ready for a recording on a new patient.
- In case you typed in wrong patient information, the information can be edited by clicking on "Edit patient information".

## 5. Order a Holter report

Cortrium Apex allows you to send recordings for analysis and download the Holter report as soon as it is available. When a device is returned with a recording, click on the Open device button and now you can select your next steps using the "Actions" button. By default, you have the following two options:

- Order Holter report
- Upload and export as EDF

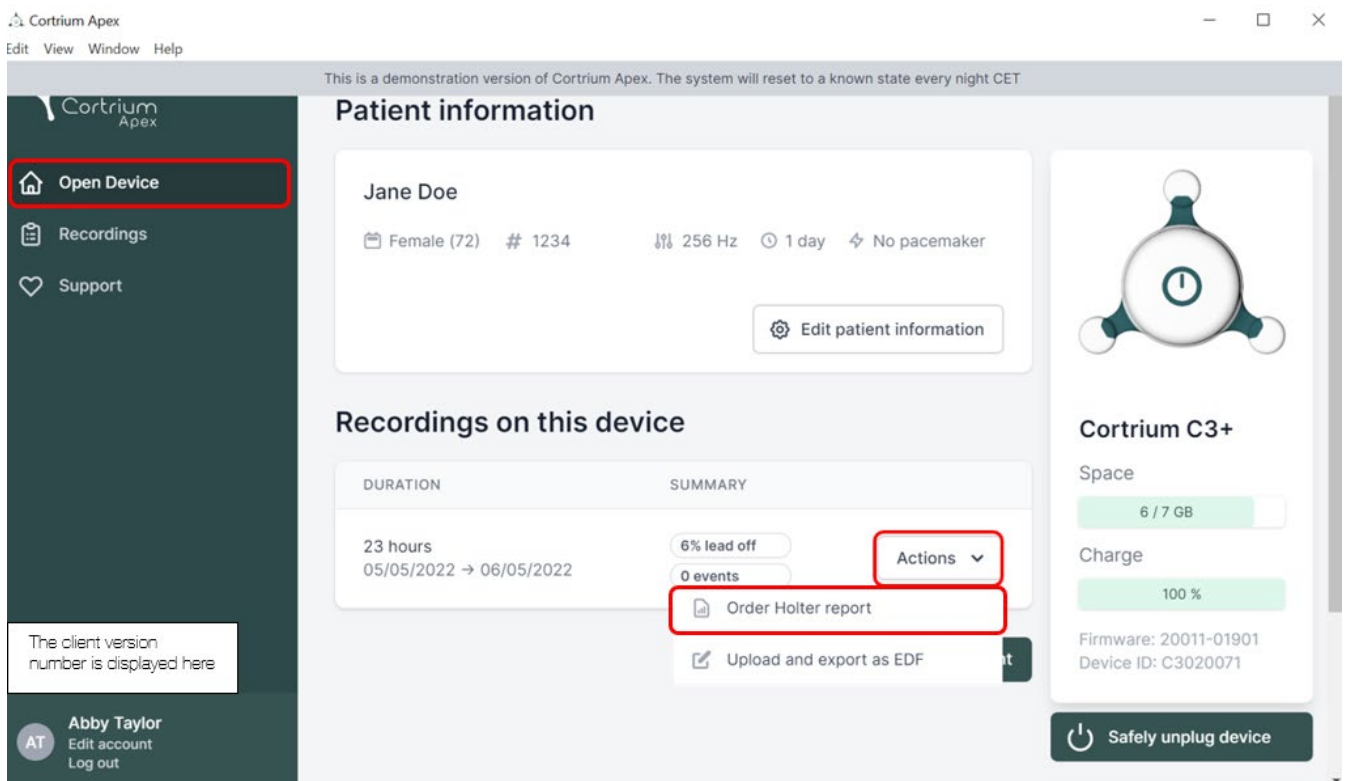


The screenshot shows the Cortrium Apex web application interface. The top bar includes the Cortrium Apex logo and a notification: "This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET". The left sidebar contains navigation links: "Open Device" (highlighted with a red box), "Recordings", and "Support". The main content area is divided into sections: "Patient information" for Jane Doe (Female, 72, # 1234, 256 Hz, 1 day, No pacemaker) with an "Edit patient information" button; "Recordings on this device" showing a recording of 23 hours from 05/05/2022 to 06/05/2022, with 6% lead off and 0 events. The "Actions" dropdown menu (highlighted with a red box) is open, showing "Order Holter report" (highlighted with a red box) and "Upload and export as EDF". The right sidebar displays device information for the Cortrium C3+, including Space (6 / 7 GB), Charge (100 %), Firmware (20011-01901), and Device ID (C3020071), along with a "Safely unplug device" button. A user profile for Abby Taylor is visible in the bottom left corner.

To order the Holter report, click on the "Actions" button and then on Order Holter report. The recording is now uploading.

Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET



- Open Device
- Recordings
- Support

### Patient information

**Jane Doe**

Female (72) # 1234 256 Hz 1 day No pacemaker

Edit patient information

### Recordings on this device

DURATION	SUMMARY	
23 hours 05/05/2022 → 06/05/2022	6% lead off 0 events	Actions

**Cortrium C3+**

Space: 6 / 7 GB

Charge: 100 %

Firmware: 20011-01901

The client version number is displayed here

Abby Taylor  
Edit account  
Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 12%

After uploading the recording, the Holter report will be made available when analysed. The device can already be prepared for a new patient as soon as the recording has been uploaded.

Depending on your account settings you will be informed via email when the Holter report is ready to view. Apex also presents newly completed reports with a red dot for easy identification.

The next section explains how to view the Holter report as soon it is analysed and how to view the entire recording right after upload.




## 6. View Holter reports and the entire recording

In Cortrium Apex you can view the Holter reports as PDF and you can view the entire recording right after uploading it for analysis. How to view the Holter reports is described below and how to use the "Recording viewer" is explained in section 6.2.

### 6.1 View Holter reports

Once a recording has been analysed, you can view the Holter report as a PDF.

To do this, go to the menu item "Recordings" and click on the respective patient. You can then view the PDF report in the last line under Holter report. The Holter report will open in your web browser.

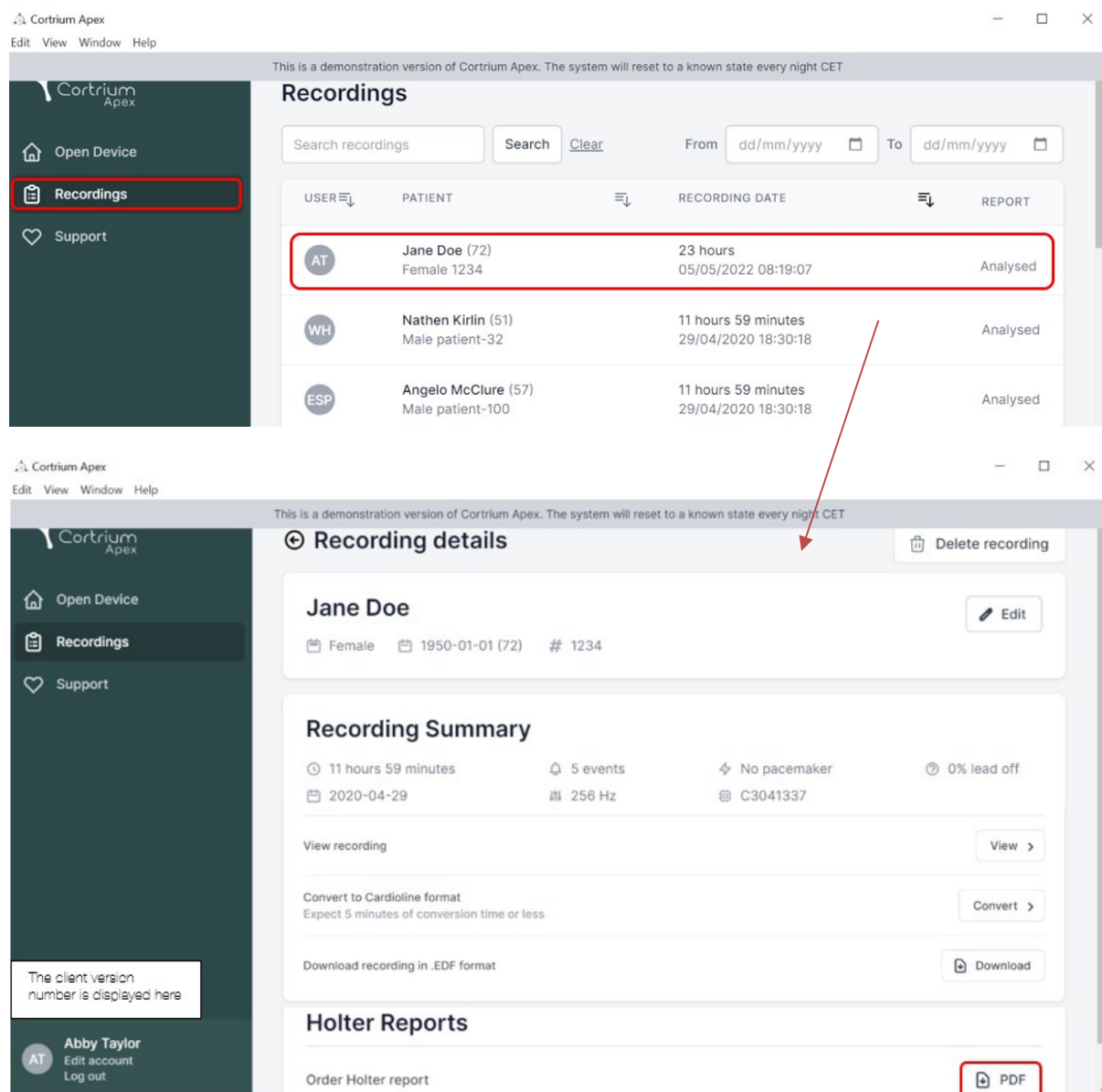

The screenshot shows the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', 'Window', and 'Help'. The left sidebar contains 'Open Device', 'Recordings' (highlighted with a red box), and 'Support'. The main content area displays a table of recordings. The first row, for Jane Doe (72), is highlighted with a red box. A red arrow points from this row to the 'Recording details' page below. The 'Recording details' page shows patient information (Jane Doe, Female, 1950-01-01 (72), # 1234) and a 'Recording Summary' section. The summary includes: 11 hours 59 minutes, 5 events, No pacemaker, 0% lead off, 2020-04-29, 256 Hz, and C3041337. Below the summary are buttons for 'View recording', 'Convert to Cardiolite format', and 'Download recording in .EDF format'. The 'Holter Reports' section at the bottom has a 'PDF' button highlighted with a red box. A text box in the bottom left corner states: 'The client version number is displayed here'.

USER	PATIENT	RECORDING DATE	REPORT
AT	Jane Doe (72) Female 1234	23 hours 05/05/2022 08:19:07	Analysed
WH	Nathen Kirlin (51) Male patient-32	11 hours 59 minutes 29/04/2020 18:30:18	Analysed
ESP	Angelo McClure (57) Male patient-100	11 hours 59 minutes 29/04/2020 18:30:18	Analysed

**Recording details**

**Jane Doe** [Edit]

Female 1950-01-01 (72) # 1234

**Recording Summary**

- 11 hours 59 minutes
- 5 events
- No pacemaker
- 0% lead off
- 2020-04-29
- 256 Hz
- C3041337

View recording [View]

Convert to Cardiolite format  
Expect 5 minutes of conversion time or less [Convert]

Download recording in .EDF format [Download]

**Holter Reports**


Order Holter report [PDF]

The client version number is displayed here

Abby Taylor  
Edit account  
Log out

## 6.2 Recording viewer

To view the entire recording after upload, click on recordings and select the patient whose recording you want to view (see first screenshot in section 6.1). Now you can use the Recording viewer by pressing the "View" button.

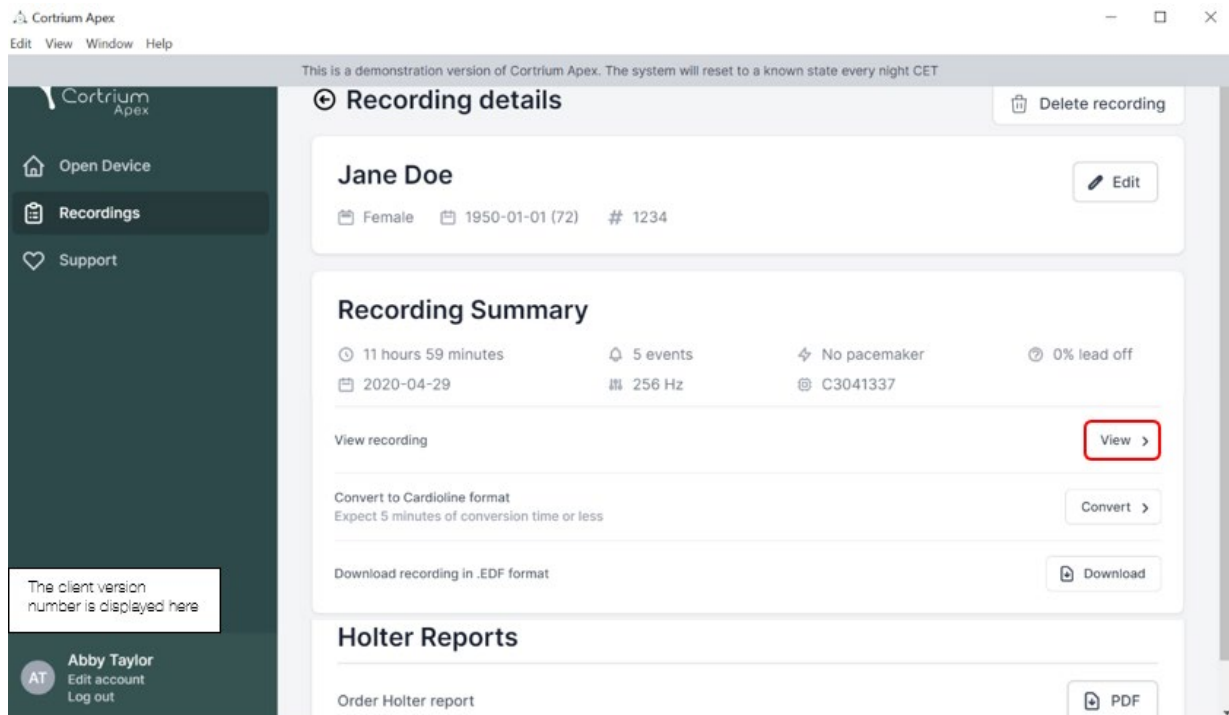


The recording viewer opens.

- The viewer loads 1 minute of data, and shows 7 seconds of ECG at a time.
- With the highlighted interactive features, you can choose the day, hour, minute, and second you would like to view in the recording. Select the time and click on the search icon to confirm the setting.
- By holding and dragging the light blue bar on the bottom of the viewer you can view the 60 seconds surrounding the chosen timepoint.
- By clicking on the navigation arrows or dragging the view to the edge, you can move forwards and backwards in the recording.
- Additionally, if the signal amplitude is low, you can increase it by using the zoom functionality.

## Recording viewer

X

2023-03-13 11:54 → 2023-03-14 11:55 1 day

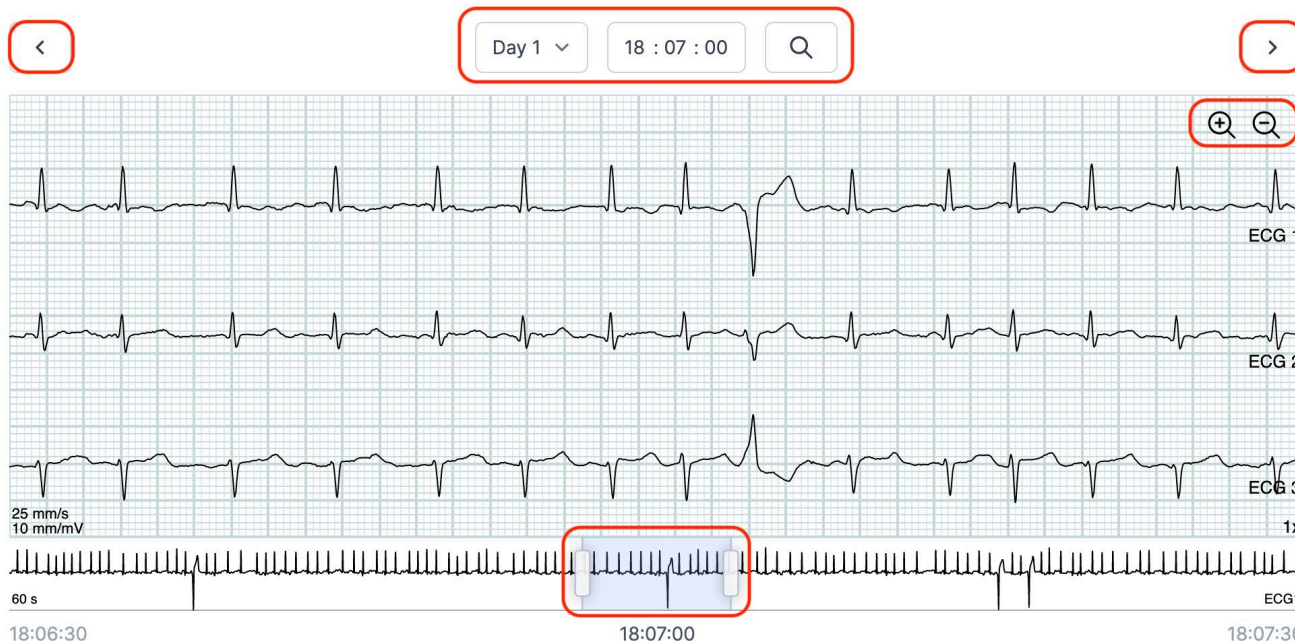

To close the Recording viewer, click on the "x" in the top right corner.

## 6.3 Summary editor

After receiving your report, you have the option for editing the summary if you have specific wording or information you would like to include.

To do this, click on recordings and select the patient whose report summary you would like to edit.

Now you can use the Summary editor by pressing the "Edit report" button in the last line under Holter report.


## 7. Edit patient details and view details of recordings

After selecting the menu item "Recordings", you can click on a recording to open a summary with the related details.

The summary of the recordings contains details such as:

- Duration of the recording
- Start date and time
- Lead off, i.e., periods when the electrodes were not in contact with the body

- Events, when the patient pressed the button to indicate symptoms or an event
- Sampling frequency of the ECG signal
- Information about pacemaker if noted when the device was prepared for a new patient

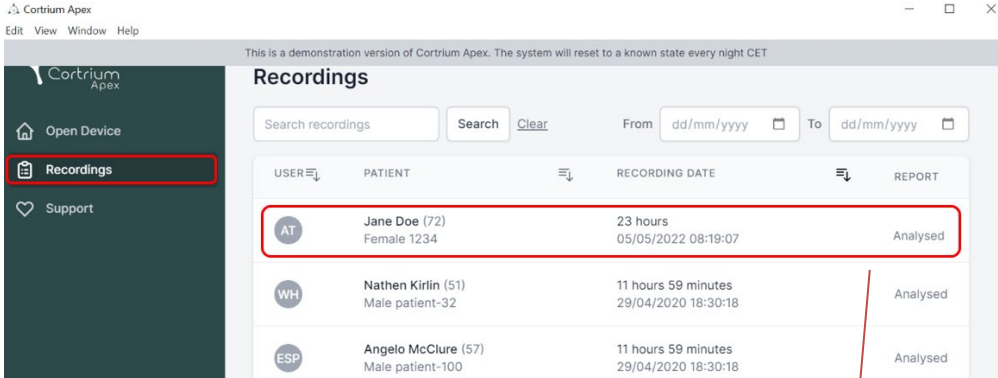


This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

### Recordings

Search recordings  Search [Clear](#) From  To

USER	PATIENT	RECORDING DATE	REPORT
AT	Jane Doe (72) Female 1234	23 hours 05/05/2022 08:19:07	Analysed
WH	Nathen Kirilin (51) Male patient-32	11 hours 59 minutes 29/04/2020 18:30:18	Analysed
ESP	Angelo McClure (57) Male patient-100	11 hours 59 minutes 29/04/2020 18:30:18	Analysed

The patient information can be edited by clicking on the "Edit" button.

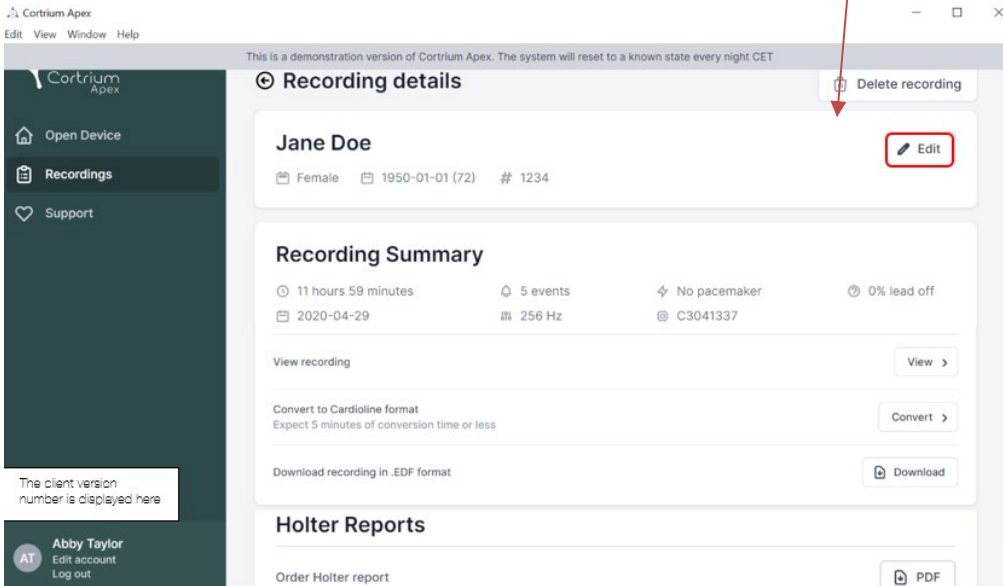

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

### Recording details

[Delete recording](#)

**Jane Doe**  
Female 1950-01-01 (72) # 1234 [Edit](#)

#### Recording Summary

11 hours 59 minutes 5 events No pacemaker 0% lead off  
2020-04-29 256 Hz C3041337

View recording [View >](#)


Convert to Cardiolite format  
Expect 5 minutes of conversion time or less [Convert >](#)

Download recording in .EDF format [Download](#)

#### Holter Reports

Order Holter report [PDF](#)

The adjustments can be saved by clicking on the "Save" button.

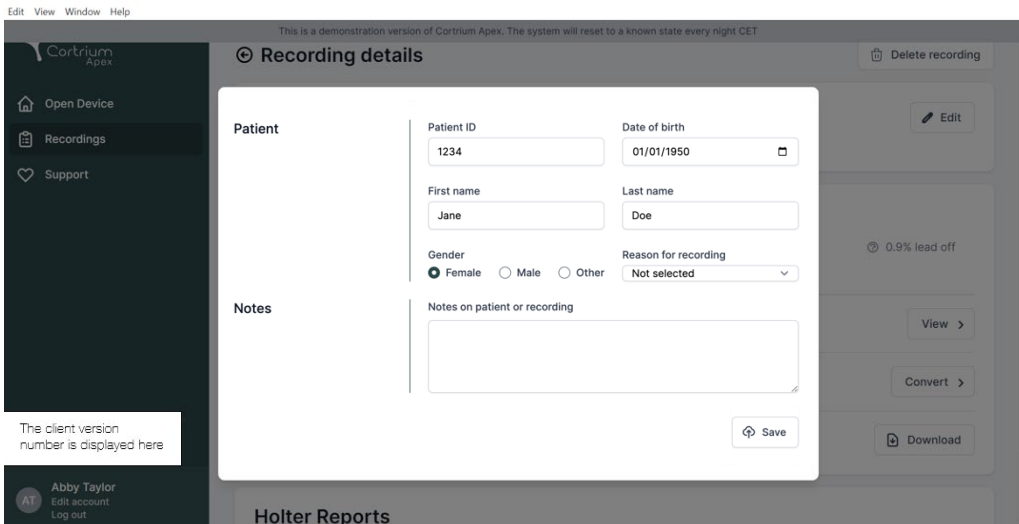

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

### Recording details

[Delete recording](#)

[Edit](#)

**Patient**

Patient ID  1234 Date of birth  01/01/1950

First name  Jane Last name  Doe

Gender ☒ Female ☐ Male ☐ Other Reason for recording  Not selected

**Notes**

Notes on patient or recording

[Save](#)

0.9% lead off

[View >](#)

[Convert >](#)


[Download](#)

#### Holter Reports

## 8. Delete a recording

To delete a recording, open the “Recordings” section and click on the recording you want to delete.

Now click on “Delete recording” in the top right corner. After typing in the patient name the deletion must be confirmed by pressing “Delete”.

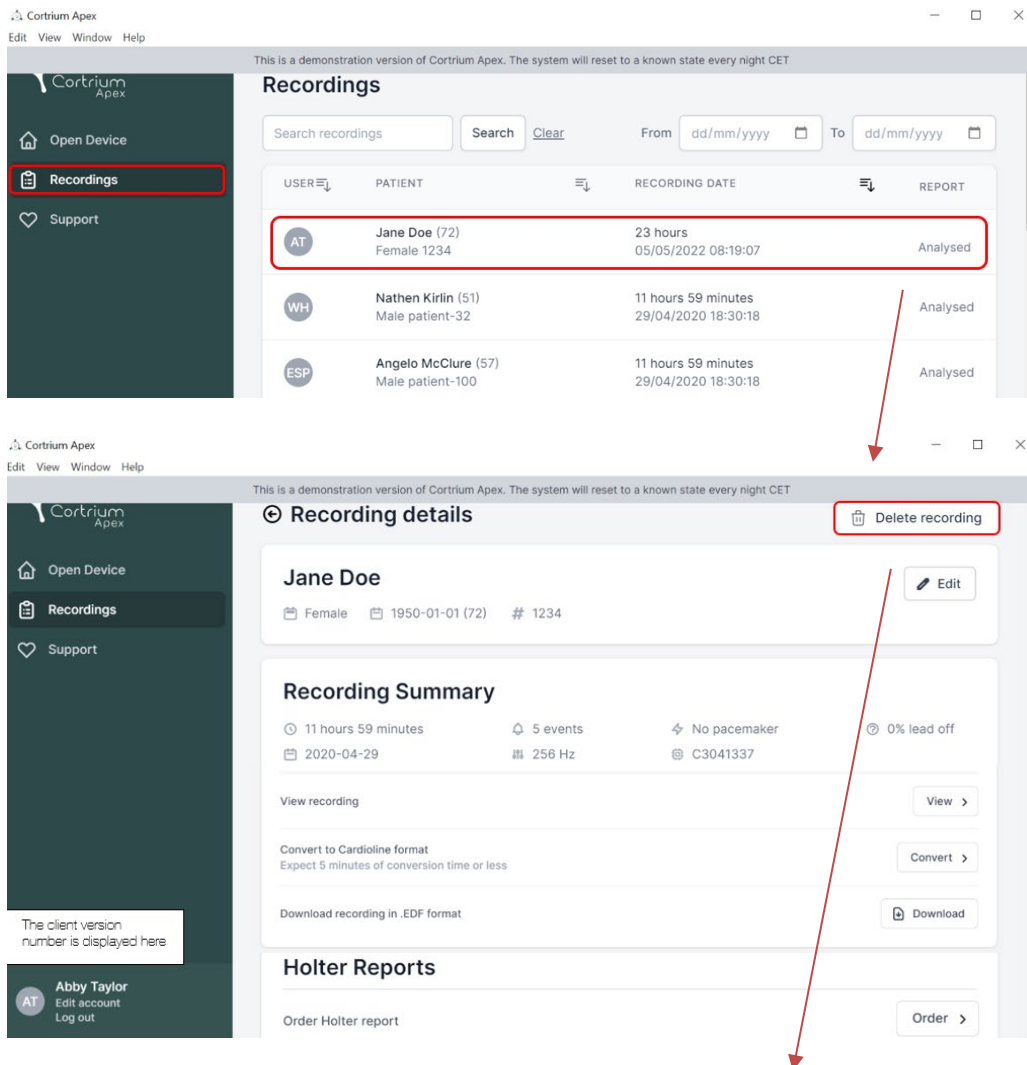

The screenshot shows the Cortrium Apex interface. The top navigation bar includes 'Open Device', 'Recordings' (highlighted with a red box), and 'Support'. The main content area displays a table of recordings. The first row, for Jane Doe (72), is highlighted with a red box. A red arrow points from this row to the 'Delete recording' button in the top right corner of the 'Recording details' view. The 'Recording details' view for Jane Doe shows a 'Delete recording' button (highlighted with a red box) and an 'Edit' button. Below the 'Recording Summary' section, there are buttons for 'View recording', 'Convert to Cardiolite format', and 'Download recording in .EDF format'. At the bottom, there is a 'Holter Reports' section with an 'Order Holter report' button. A red arrow points from the 'Delete recording' button to the 'Delete' confirmation button in the bottom right corner of the 'Delete recording' dialog box.

USER	PATIENT	RECORDING DATE	REPORT
AT	Jane Doe (72) Female 1234	23 hours 05/05/2022 08:19:07	Analysed
WH	Nathan Kirilin (51) Male patient-32	11 hours 59 minutes 29/04/2020 18:30:18	Analysed
ESP	Angelo McClure (57) Male patient-100	11 hours 59 minutes 29/04/2020 18:30:18	Analysed

**Recording details**

**Jane Doe**  
Female 1950-01-01 (72) # 1234

**Recording Summary**

- 11 hours 59 minutes
- 5 events
- No pacemaker
- 0% lead off
- 2020-04-29
- 256 Hz
- C3041337

**Holter Reports**

Order Holter report

Edit View Window Help

Cortrium  
Apex

Open Device

Recordings

Support

The client version number is displayed here

Abby Taylor  
Edit account  
Log out

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET

### Recording details

Jane Doe

Female 1940-01-01 (82) # 1234

Edit

Confirm delete

You are removing Jane Doe

To confirm delete, please write 'Jane Doe' in the input field

Jane Doe

Cancel Delete

0.9% lead off

View >

Convert >

Download

Convert to Cardiolite format

Expect 5 minutes of conversion time or less

Download recording in .EDF format

Holter Reports


## 9. Download EDF file

The EDF file can be downloaded before ordering the Holter report or after. The two different cases are described in the next sections.

### 9.1 How to download EDF file before ordering the Holter report

Follow these steps to convert a recording to EDF format:

- Connect the C3+ Holter monitor to the computer and press the “Open Device” button in the left menu.
- Click on the "Actions" button and select "Upload and export as EDF".

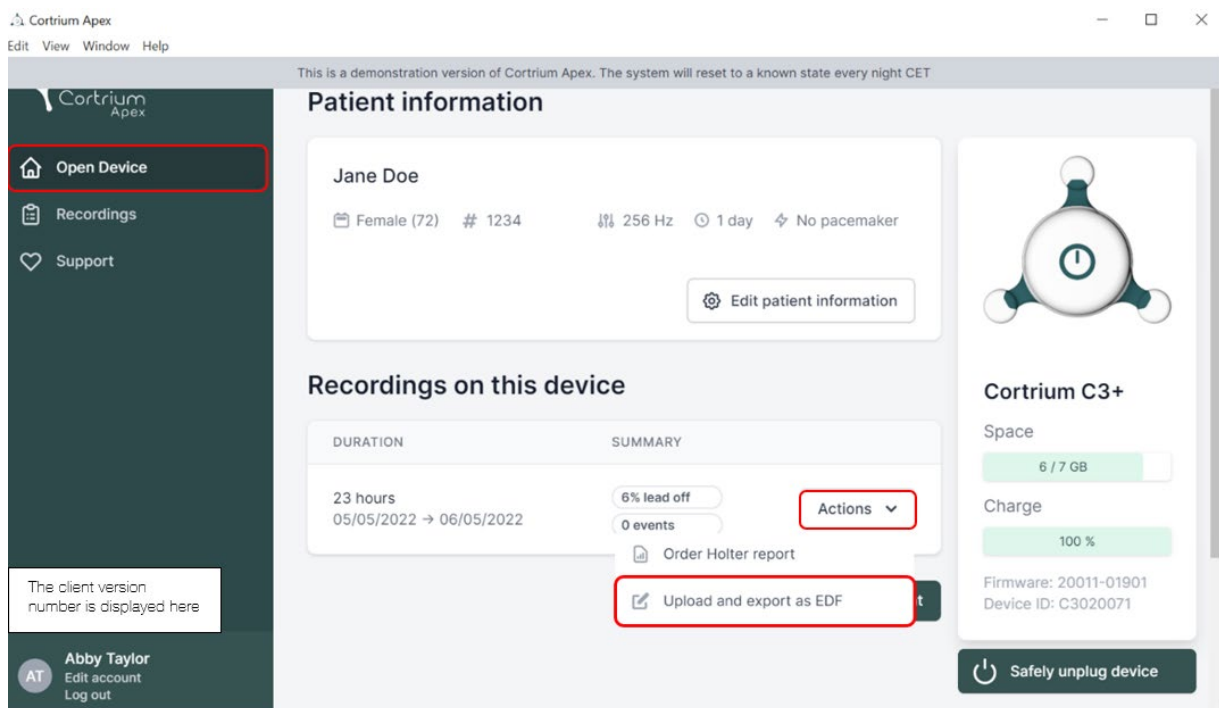


- The recording is transferred to Apex.
- When the transfer is complete, the "Actions" button changes to "View".



Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET



- Open Device
- Recordings
- Support

### Patient information


Jane Doe

Female (72) # 1234 256 Hz 1 day No pacemaker

Edit patient information

### Recordings on this device

DURATION	SUMMARY
23 hours 05/05/2022 → 06/05/2022	6% lead off 0 events Actions



**Cortrium C3+**

Space  
6 / 7 GB

Charge  
100 %

Firmware: 20011-01901  
Device ID: C3020071

The client version number is displayed here


Abby Taylor  
Edit account  
Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 12%

Edit View Window Help

This is a demonstration version of Cortrium Apex. The system will reset to a known state every night CET



- Open Device
- Recordings
- Support

### Patient information


Jane Doe

Female (72) # 1234 256 Hz 1 day No pacemaker

Edit patient information

### Recordings on this device

DURATION	SUMMARY
23 hours 05/05/2022 → 06/05/2022	6% lead off 0 events View



**Cortrium C3+**

Space  
6 / 7 GB

Charge  
100 %

Firmware: 20011-01901  
Device ID: C3020071

The client version number is displayed here

Abby Taylor  
Edit account  
Log out


Erase all and prepare for new patient

Safely unplug device

- Click on the "View" button to see the recording overview.
- The sub-item "Download report in EDF format" shows the following information:
  - While the recording is being converted, the progress is displayed



- When the EDF file is ready, click the "Download" button to save the EDF file on your computer

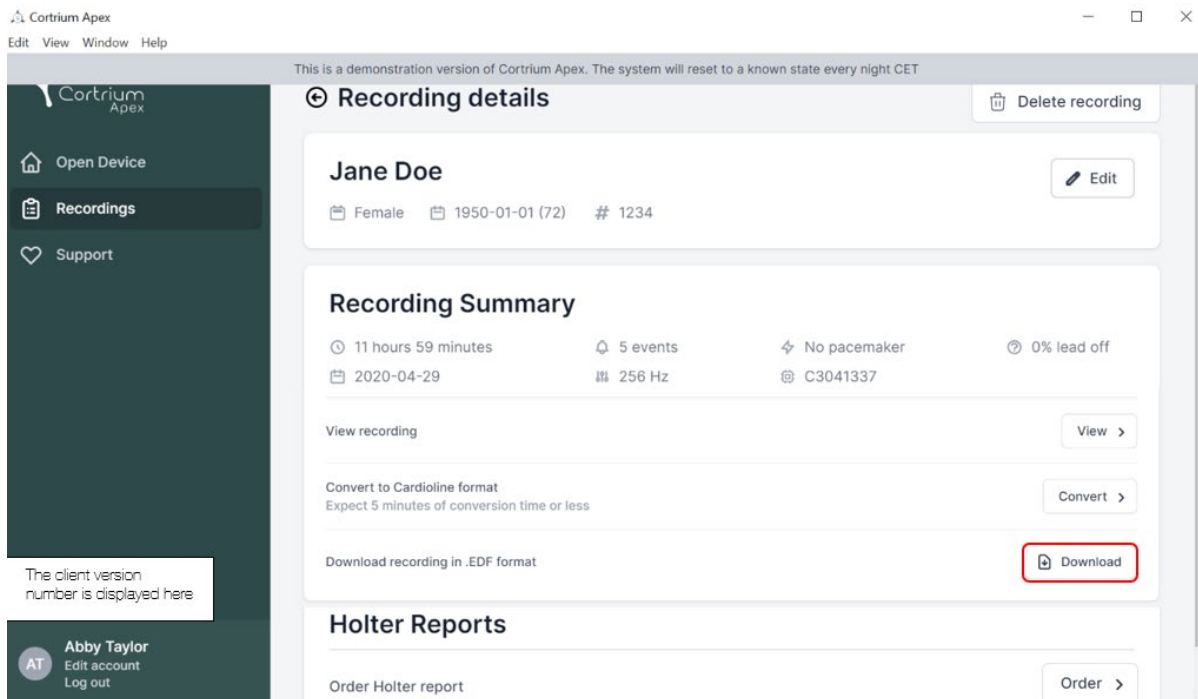


## 9.2 How to download EDF file after ordering a report

If you have just ordered a report of a recording and now want to download the recording in EDF format, proceed as follows:

- When the device is connected, click on the menu item "Open device", then click on the "View" button next to the recording.
- In the "Recording details" screen, click on "Download" to download the EDF file.

If you want to download a previous recording in EDF format:

- click on the menu item "Recordings"
- click on the recording you want to download in EDF format
- and then click on "Download" as illustrated below

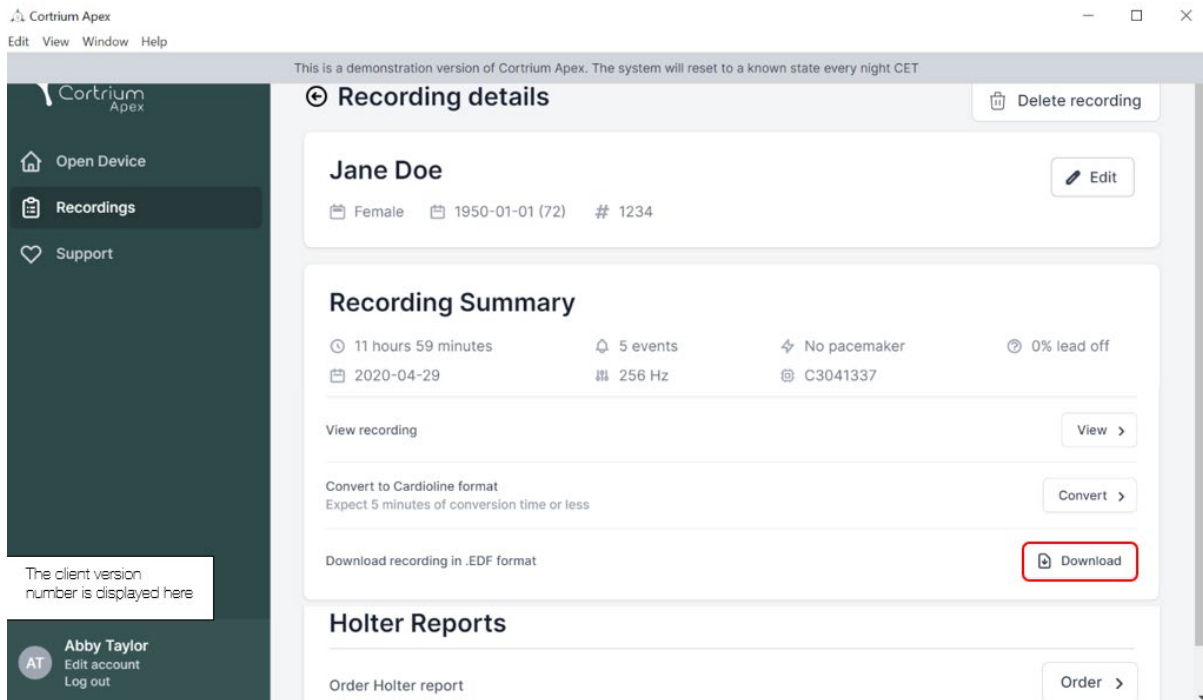

## 10. Log out of Cortrium Apex

It is important to remember to log out of the application by clicking on the "Log out" button in the bottom left corner.

Cortrium Apex will automatically log out of your user account after an extended period of inactivity.

If your institution has multiple users with separate accounts for Cortrium Apex, but the users are working from the same computer, please ensure you log out of Cortrium Apex when you have finished your tasks.

## 11. Software-Updates

When started, the application automatically checks for available updates and installs them immediately.

## 12. Minimum technical requirements

The minimum requirements for the operation of Cortrium Apex are:

- Windows 10 or MacOS Catalina (Intel).
- 1280 x 960 Pixel screen resolution.
- 4 GB RAM.

- 5 GB free hard disk space.
- Internet connection.

Cortrium Apex requires an internet connection. The upload speed of the connection determines the time it takes to upload a recording when you order a report or convert the recording to another format. The download speed determines the time it takes to download a converted recording.

Downloading and viewing a report usually takes only a few seconds on a 10 Mbps internet connection.

## 13. Troubleshooting

### 13.1 “The device seems to be set up for a patient...”

If you see the warning: "*Warning. The device appears to be set up for a patient, but we cannot find the patient data in the system. Please set up a patient for the records*",

simply click the "Add Patient Information" button to set up the device for use with Cortrium Apex. This will not remove the records on the device.

### 13.2 Recording summary shows "n/a lead off" or "n/a events"

If the recording summary shows "n/a lead off" or "n/a events", this is because the firmware on the connected C3+ unit is a version that does not support immediate lead off or event summary. However, the recording is still valid.

### 13.3 The internet connection was interrupted.

If you see the message "Cortrium Apex is having trouble accessing the Internet", your internet connection may have been interrupted or lost. Please contact your facility's IT support or your internet service provider for assistance in restoring your internet connection.

- Cortrium Apex automatically tries to connect to the Internet again. A message indicates when the next connection attempt will be made
- If there is no internet connection when Cortrium Apex is started, a message will be displayed indicating that you appear to be offline

- If you were transferring a recording when the internet connection was lost, you may need to restart the transfer when the connection is restored

### 13.4 Apex is newly installed but does not open

This issue may occur due to a firewall blocking the Cortrium Apex application. In case Apex does not start due to a firewall, please ensure that the URL links in section 16 “Infrastructure statement” are unblocked or “whitelisted” in the firewall. Please contact your facility’s IT for support.

## 14. Security information

### User management

- Every user of Cortrium Apex must use their own account.  
You can create and delete as many user accounts as you need at <https://apex.cortrium.com>  
It is important that you are able to determine and regulate who has access to the sensitive data of your patients.

### Passwords

- Ensure that you are using a safe password
- Do not share your login and/or password with others
- Avoid using the same password for multiple accounts  
Re-using the same password could result in that an adversary may be able to gain access to all your systems and computers, if just one password is compromised.
- Remember to log out of Cortrium Apex  
If you remain logged in, other users actions in the system will appear to be yours

### Encryption and additional protection

Before installing Cortrium Apex on a computer we recommend you:

- Enable disk encryption  
In case the computer is lost or stolen sensitive data on the computer will not be readable
- Enable automatic system updates  
Increases the chance that the computer stays up-to-date with security updates
- Enable built-in security software (e.g. Windows Security) if available
- Firewall, anti-malware, anti-phishing and anti-virus protection is built into Windows. We highly recommend you enable it.





## 15. Labelling and symbols

The following is a representation of the labelling and symbols found in Cortrium Apex:



**Cortrium Apex** UDI:(01)05745000379033(8012)1.15.0

*Example of label*

Symbol	Name	Requirement / note
	Date of Manufacture	ISO 15223-1
	Refer to instruction manual/booklet	ISO 7000 - 1641
	Caution sign	ISO 7000 - 0434A
	CE approved device with notifying body identification number	MDR (EU/745/2017)
	Medical Device	ISO 15223-1:2021

## 16. Infrastructure statement

Cortrium ApS declares that its product, Cortrium Apex is deployed in the Microsoft Azure cloud computing service. Some of the URLs are dependent on the location where the product is being used. The table below contains addresses used by the system:

URL	Description
<a href="https://api.apex.cortrium.com">api.apex.cortrium.com</a>	Cortrium Apex API.

<a href="http://viewer.apex.cortrium.com">viewer.apex.cortrium.com</a>	Cortrium Recording Viewer API.
<a href="http://apex.cortrium.com">apex.cortrium.com</a>	Cortrium Apex administrator interface.
<a href="http://client.apex.cortrium.com">client.apex.cortrium.com</a>	Cortrium Apex web client.
<a href="http://login.cortrium.com">login.cortrium.com</a>	Auth0 login endpoint.
<a href="http://assets.apex.cortrium.com">assets.apex.cortrium.com</a>	Assets downloads, such as Desktop Client.
<a href="http://support.cortrium.com">support.cortrium.com</a>	Support website.
<a href="http://cortriumukrecordingapex.blob.core.windows.net">cortriumukrecordingapex.blob.core.windows.net</a>	Recordings blob storage (UK South).
<a href="http://cortriumfrrecordingapex.blob.core.windows.net">cortriumfrrecordingapex.blob.core.windows.net</a>	Recordings blob storage (France Central).
<a href="http://cortriumeurecordingapex.blob.core.windows.net">cortriumeurecordingapex.blob.core.windows.net</a>	Recordings blob storage (Germany West Central).
<a href="http://cortriumaurecordingapex.blob.core.windows.net">cortriumaurecordingapex.blob.core.windows.net</a>	Recordings blob storage (Australia East).
<a href="http://rt.services.visualstudio.com">rt.services.visualstudio.com</a> <a href="http://dc.services.visualstudio.com">dc.services.visualstudio.com</a>	Used by Microsoft Azure Application Insights (telemetry).

Port 443 (HTTPS) needs to be opened for all the URL's above. The first 7 rules of this list can be simplified by whitelisting the whole Cortrium domain (\*.cortrium.com). This also reduces the risk of needing to make future adjustments, when this list changes.

