

Cortrium C3+ app

Instructions for use


1. Start Bluetooth mode of the C3+ Holter Monitor 2
2. Open the Cortrium C3+ app..... 3
3. Connect the C3+ Holter Monitor with the app..... 3


The Cortrium C3+ app is only for the visualization of the signal quality. Please note that this app does not possess the ability to analyze the signal and should not be used to make direct diagnostic evaluations. Please use the data stored on the C3+ for interpretation and analysis.


You can download the app in the Apple store and Google Play store by searching for "Cortrium C3+".

1. Start Bluetooth mode of the C3+ Holter Monitor

To use the C3+ Holter Monitor with the iOS app you need to first turn on your C3+ with a single push of the button, followed by a double push to start the Bluetooth mode. The device now flashes blue.

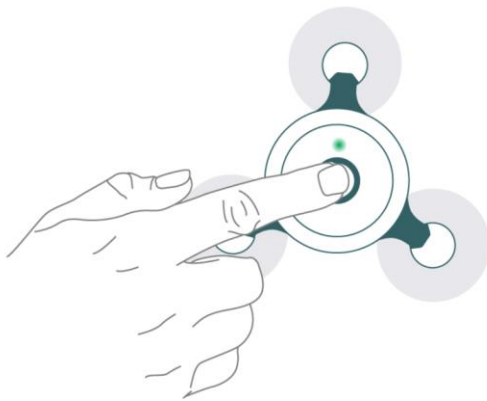


	Single press	Turn on device
---	-----------------	-------------------

	Double press	Activate Bluetooth
---	-----------------	-----------------------


2. Open the Cortrium C3+ app

Press the Cortrium C3+ app icon to open the app.

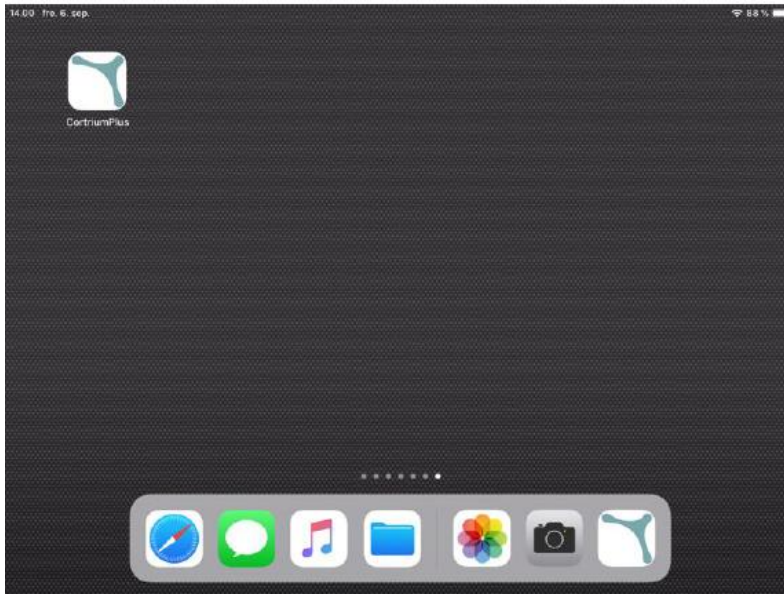

3. Connect the C3+ Holter Monitor with the app

Press the "Connect" button in the top right corner of the screen.

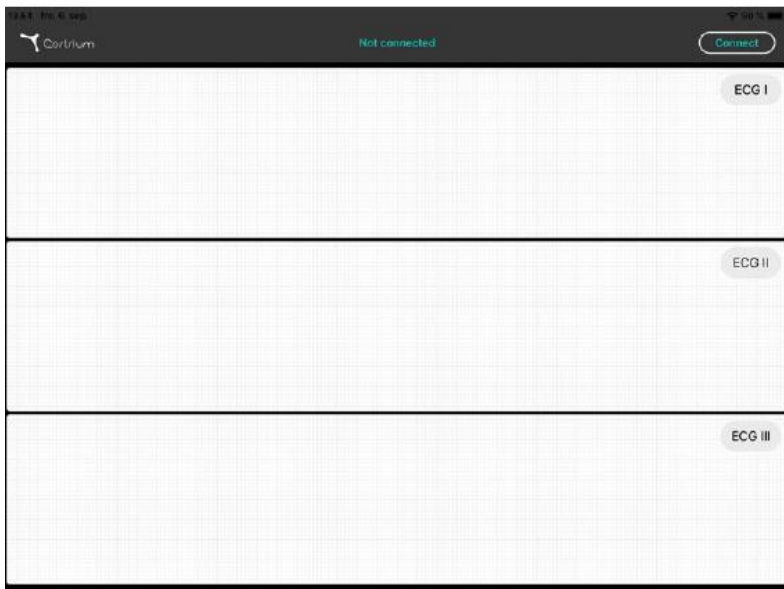

Choose from the listed C3+ devices or cancel pairing.



When the device is placed on the patient's chest the ECG will appear on the screen. To disconnect, press the button "Disconnect".



The Bluetooth mode of the C3+ Holter Monitor switches off automatically after 10 minutes. To switch off the Bluetooth mode before, double push the button of the device. The device now flashes green slowly.