# Cortrium Apex 1.20

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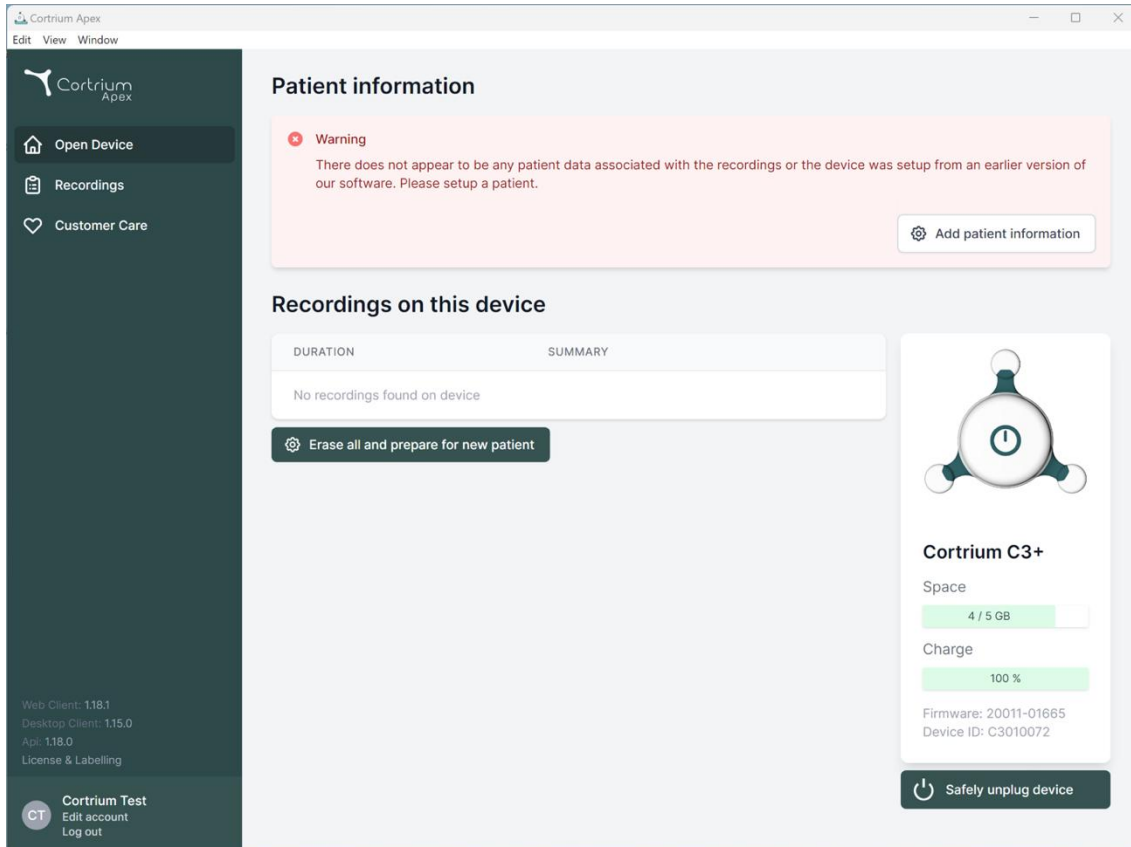
The screenshot shows the Cortrium Apex web interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, it shows version information (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, License & Labelling) and user options for 'Cortrium Test' (Edit account, Log out).

The main content area is titled 'Patient information' and features a warning message: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' Below this is a button to 'Add patient information'.

Below the warning is a section titled 'Recordings on this device' with a table header for 'DURATION' and 'SUMMARY'. The table is currently empty, displaying 'No recordings found on device'. A button below the table reads 'Erase all and prepare for new patient'.

On the right side, there is a device status card for 'Cortrium C3+'. It shows a battery icon, 'Space' usage at '4 / 5 GB', 'Charge' at '100%', and technical details: 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of the card.

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Cortrium Apex  
Edit View Window

Cortrium Apex

Open Device  
Recordings  
Customer Care

### Patient information

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960  
Date of birth: 08/28/1960

First name: Cortrium  
Last name: Test

Gender:  Female  Male  Other  
Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day  
Pacemaker: None  
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

**⚠ This will erase all recordings on the device**

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
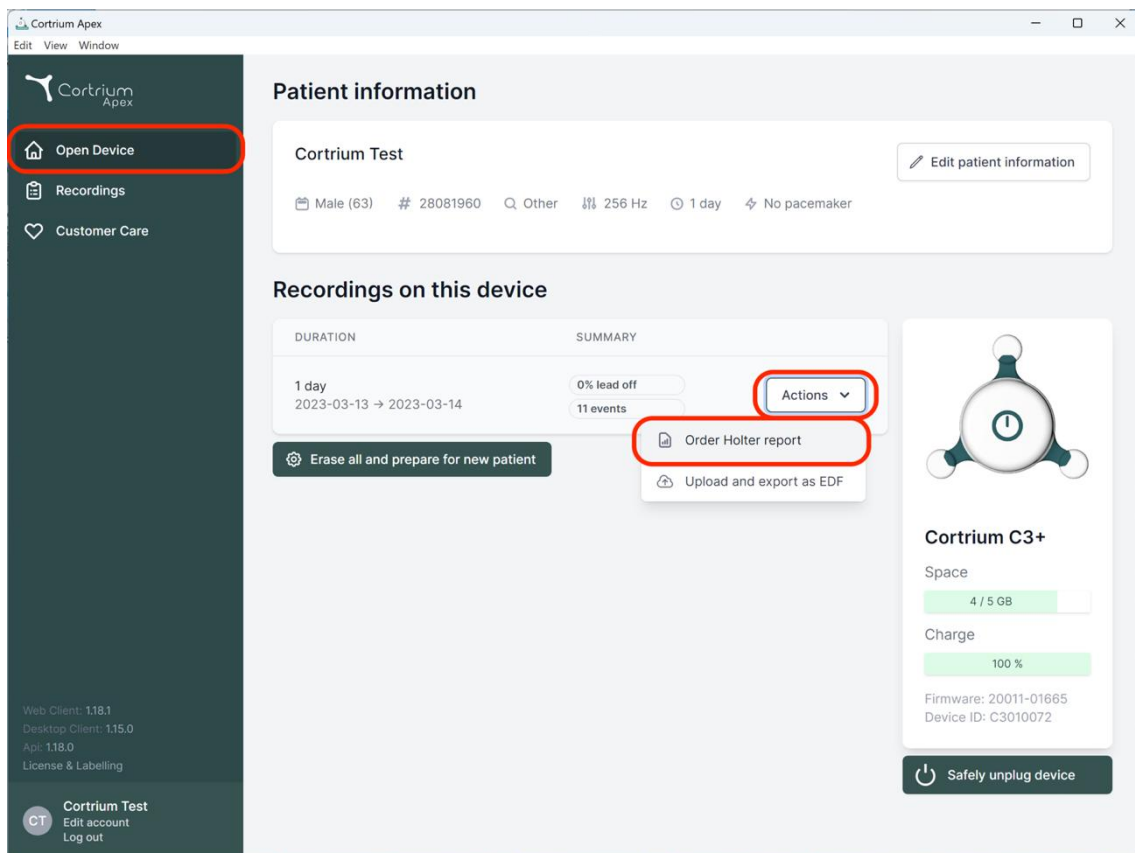
The screenshot displays the Cortrium Apex web application interface. On the left, a dark sidebar contains navigation options: 'Open Device' (highlighted with a red box), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and test parameters (Other, 256 Hz, 1 day, No pacemaker). Below this, the 'Recordings on this device' section shows a recording for '1 day' (2023-03-13 to 2023-03-14) with '0% lead off' and '11 events'. An 'Actions' dropdown menu (highlighted with a red box) is open, showing 'Order Holter report' (highlighted with a red box) and 'Upload and export as EDF'. On the right, a device status card for 'Cortrium C3+' shows 'Space: 4 / 5 GB' and 'Charge: 100%'. A 'Safely unplug device' button is at the bottom right. The footer includes version information and a user profile for 'Cortrium Test'.

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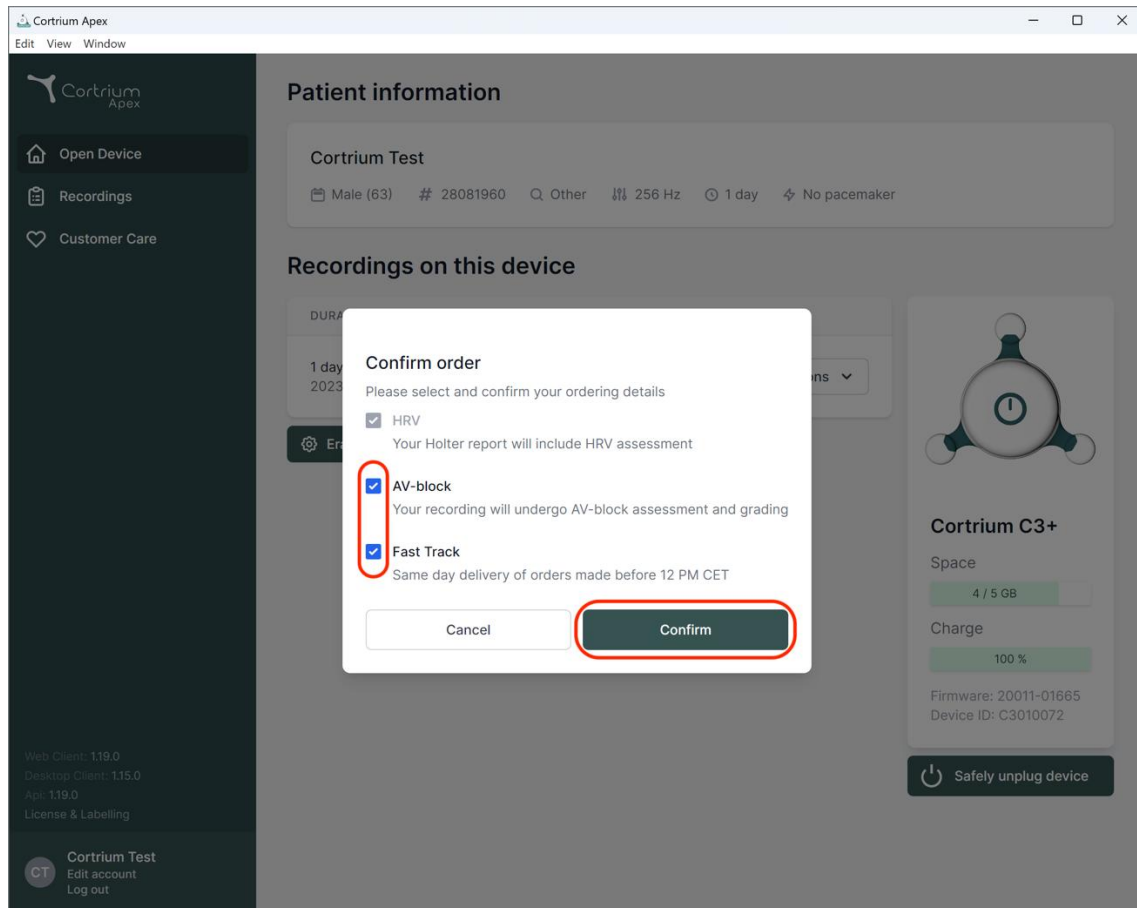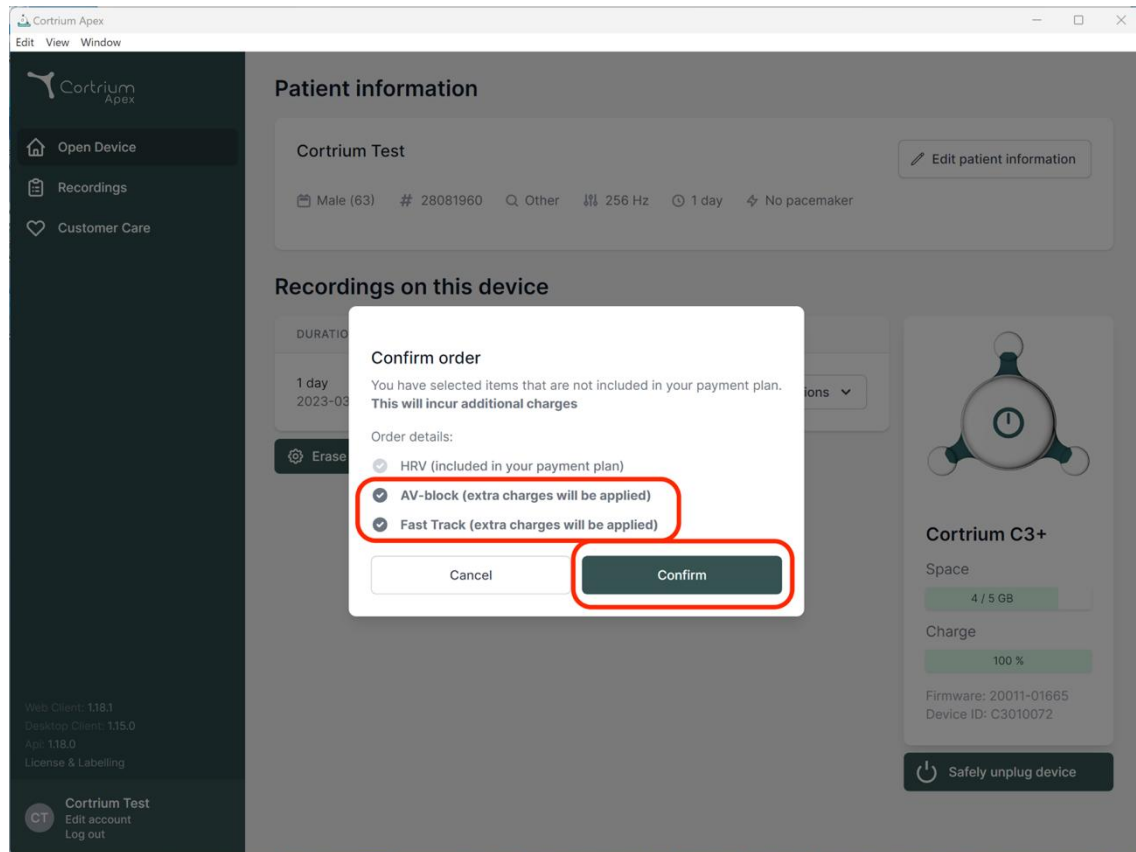
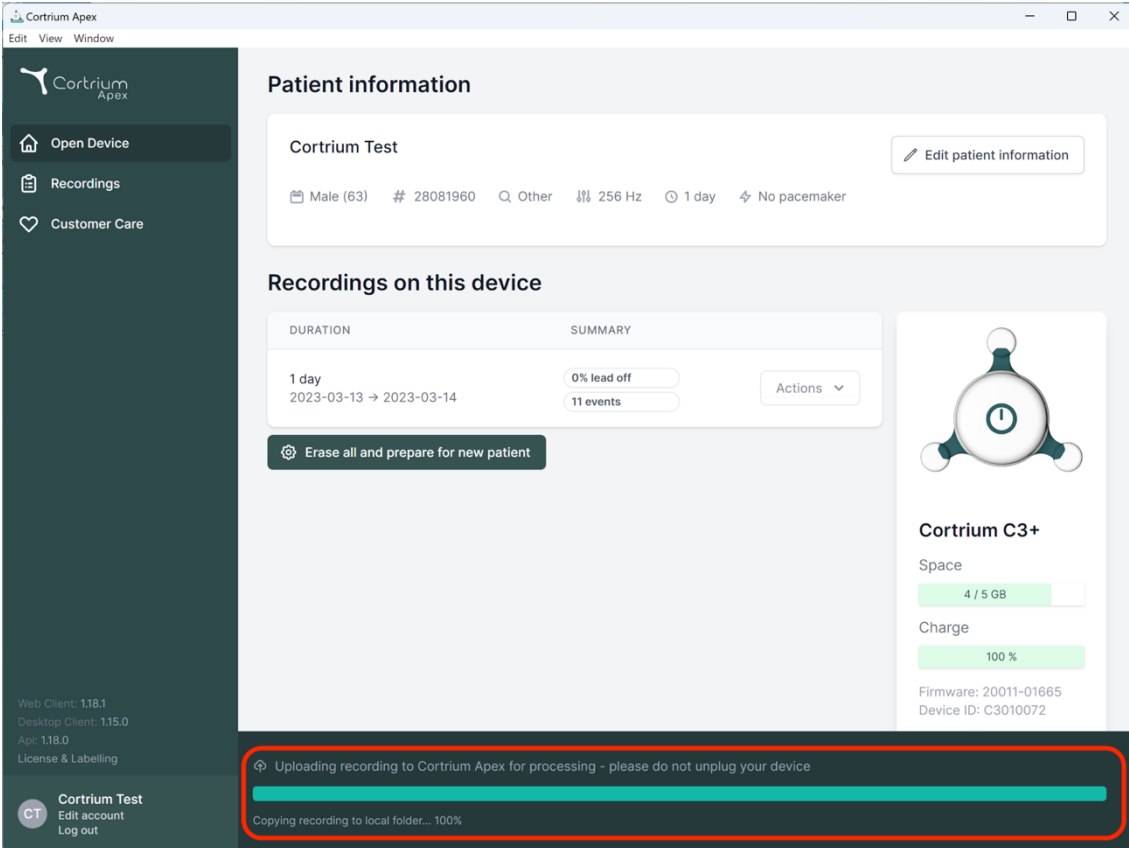
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, a 'Recordings on this device' section shows a summary of 0% lead off and 11 events. To the right, a device status card for 'Cortrium C3+' indicates 4/5 GB space and 100% charge. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. The sidebar also contains version information and user options like 'Edit account' and 'Log out'.

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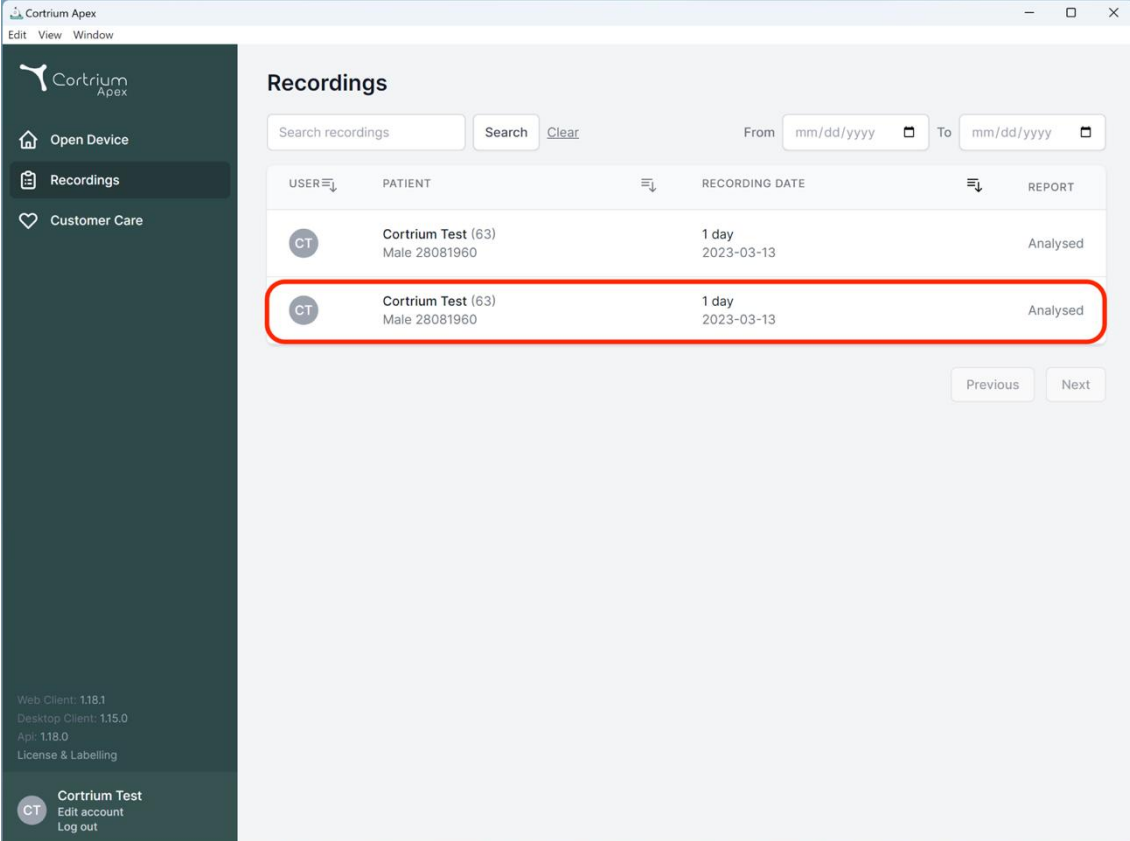
The screenshot shows the 'Recordings' page in the Cortrium Apex web application. The page has a dark sidebar on the left with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Recordings' and features a search bar with 'Search' and 'Clear' buttons, and date filters for 'From' and 'To'. Below the search bar is a table with the following data:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

The second row of the table is highlighted with a red border. At the bottom of the table, there are 'Previous' and 'Next' buttons. The sidebar also contains version information: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'Api: 1.18.0', and 'License & Labelling'. At the bottom of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

Cortrium Apex  
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1  
Desktop Client: 1.15.0  
App: 1.18.0  
License & Labelling

**CT** Cortrium Test  
Edit account  
Log out

### Recording details

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

### Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

### Recording Summary

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

### Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
 HRV  AV-block  Fast Track

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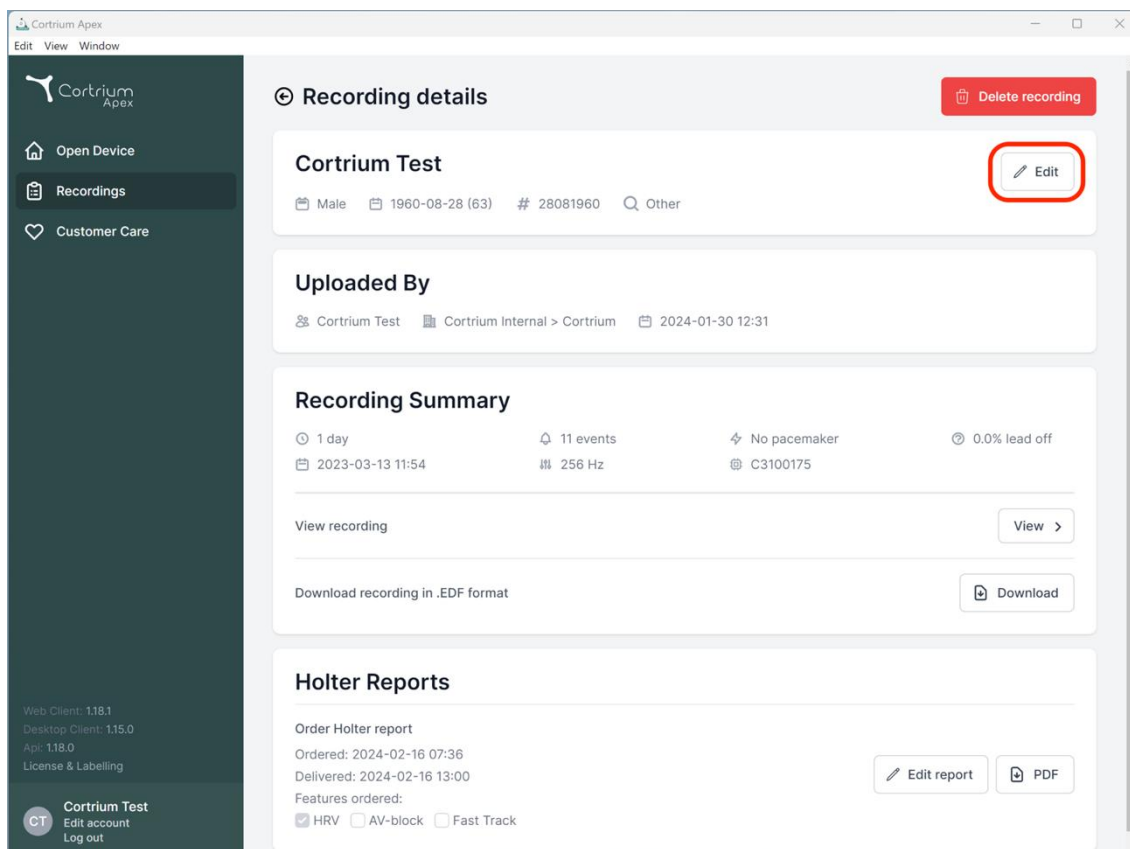
**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**  
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

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**Holter Reports**

Order Holter report  
Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00  
Features ordered:  
 HRV  AV-block  Fast Track

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
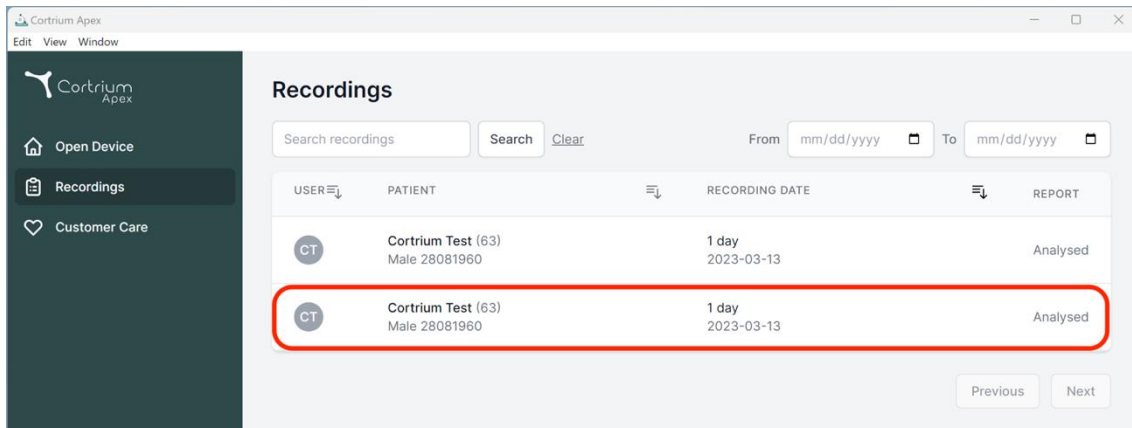
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. A table lists recordings with columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row is highlighted with a red border.

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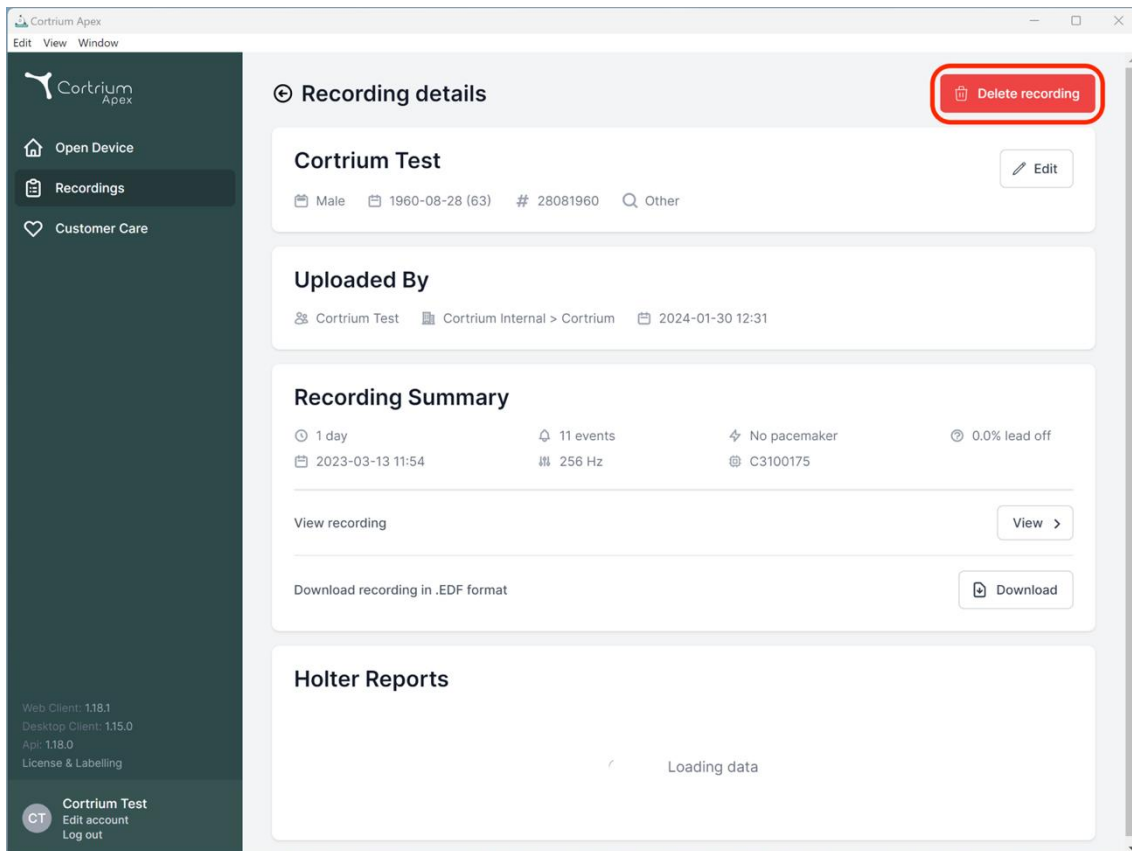The screenshot shows the 'Recording details' page for 'Cortrium Test'. A red box highlights the 'Delete recording' button in the top right corner.

**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**  
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

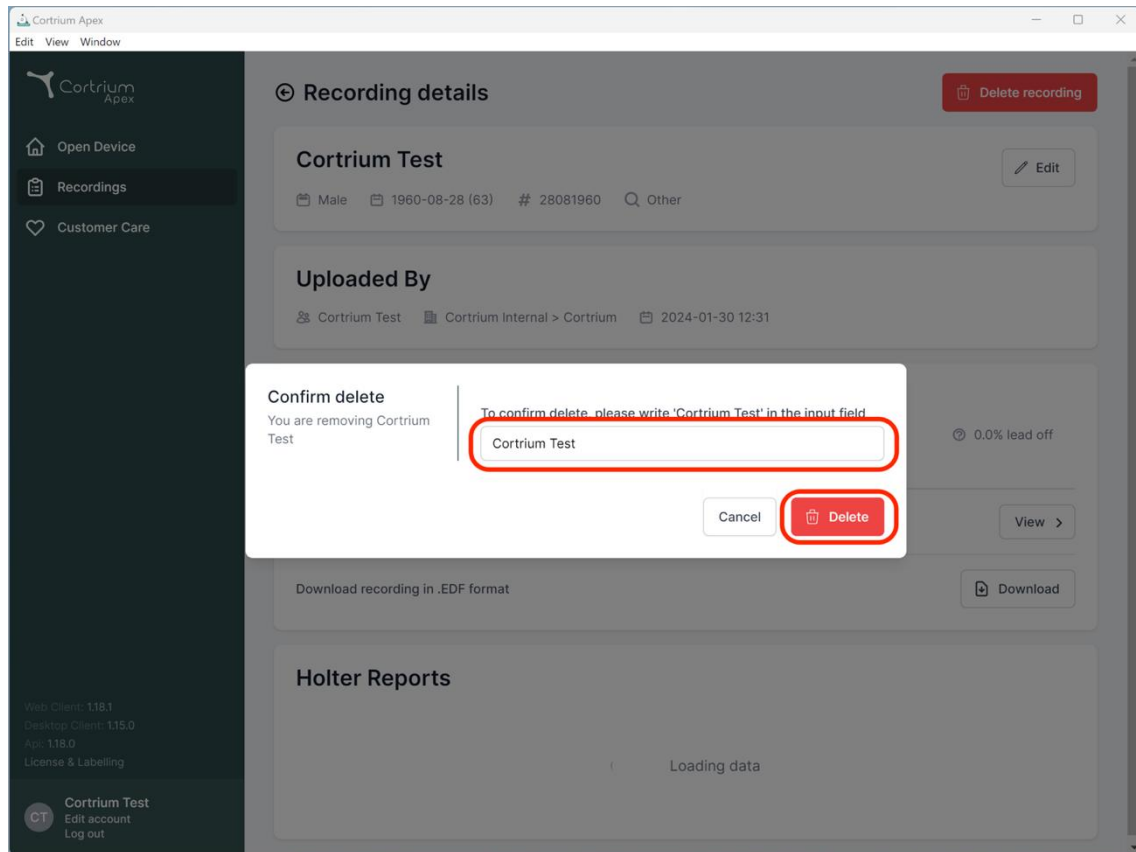
- 1 day
- 11 events
- No pacemaker
- 0.0% lead off
- 2023-03-13 11:54
- 256 Hz
- C3100175

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Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

CT Cortrium Test  
Edit account  
Log out

**Confirm delete**  
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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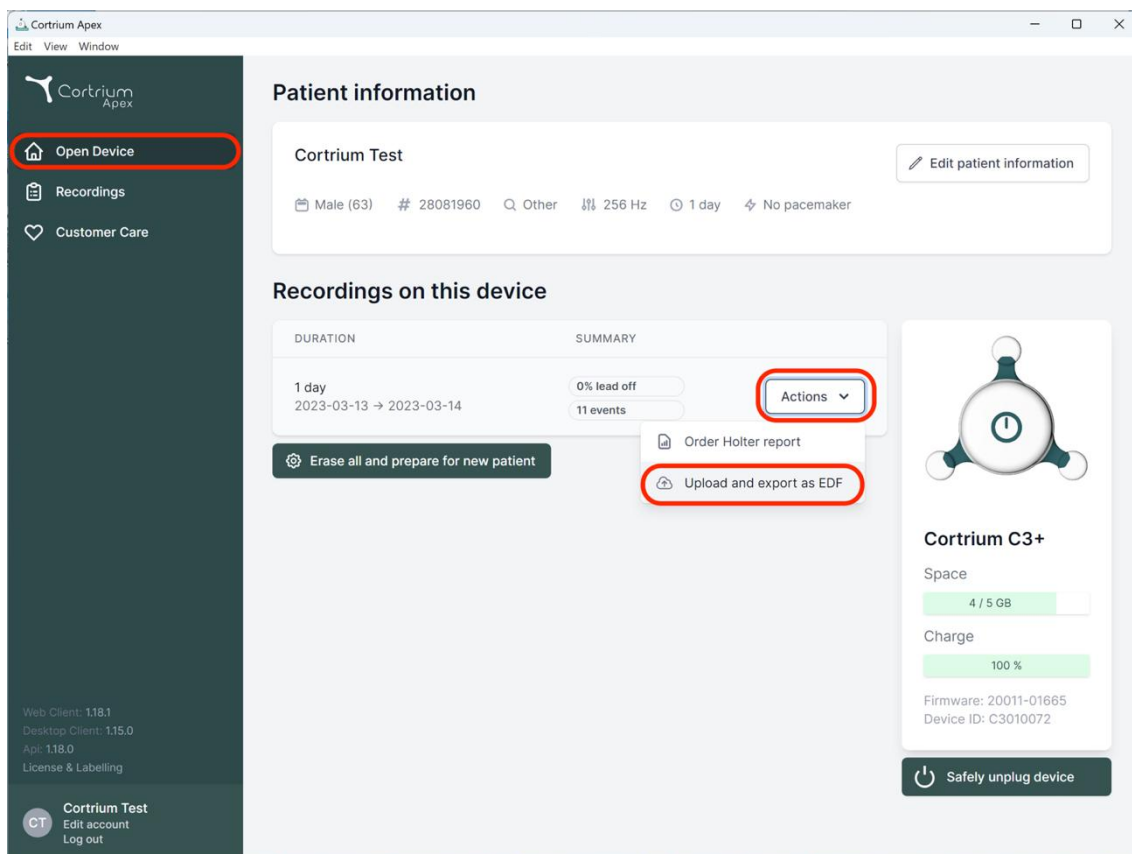
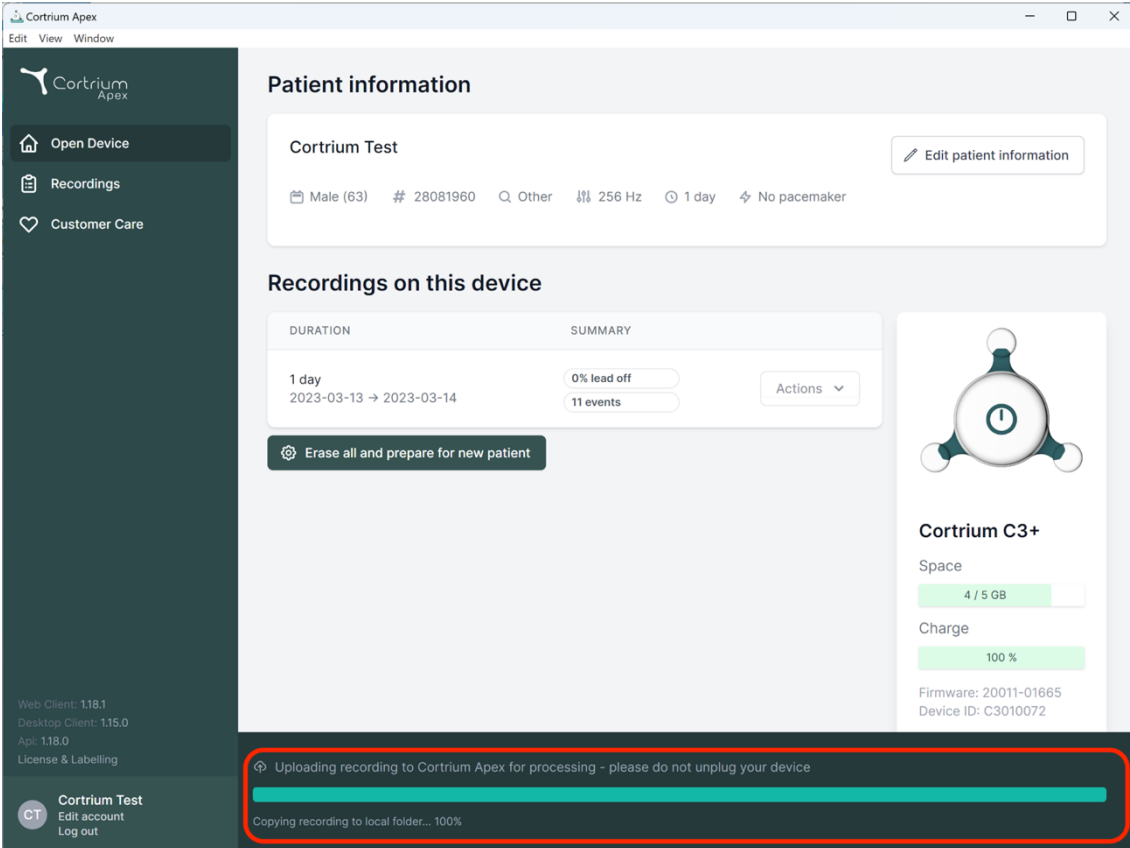
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">Actions</a>

[Erase all and prepare for new patient](#)

**Cortrium C3+**

Space: 4 / 5 GB

Charge: 100 %

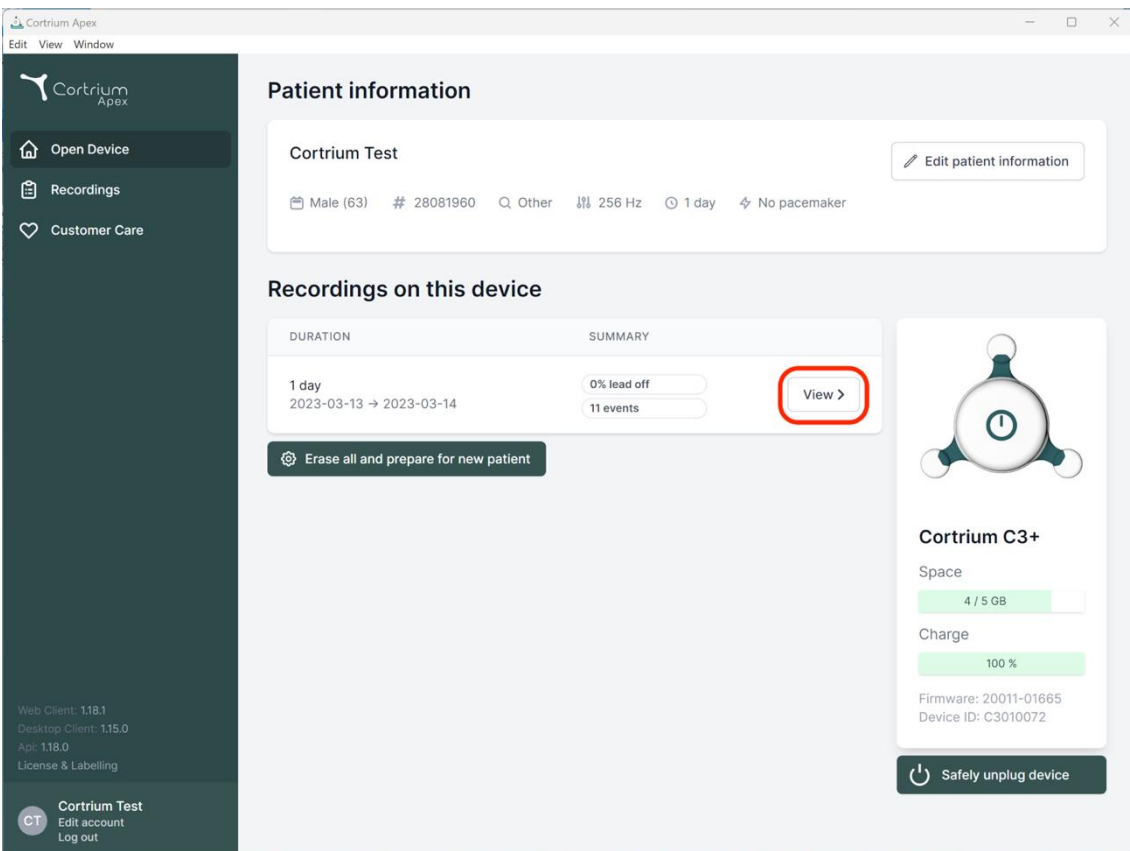
Firmware: 20011-01665  
Device ID: C3010072

Web Client: 1.18.1  
Desktop Client: 1.15.0  
App: 1.18.0  
License & Labelling

**Cortrium Test**  
CT Edit account Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

**Cortrium Apex**  
Edit View Window

**Patient information**

**Cortrium Test** [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">View &gt;</a>

[Erase all and prepare for new patient](#)

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
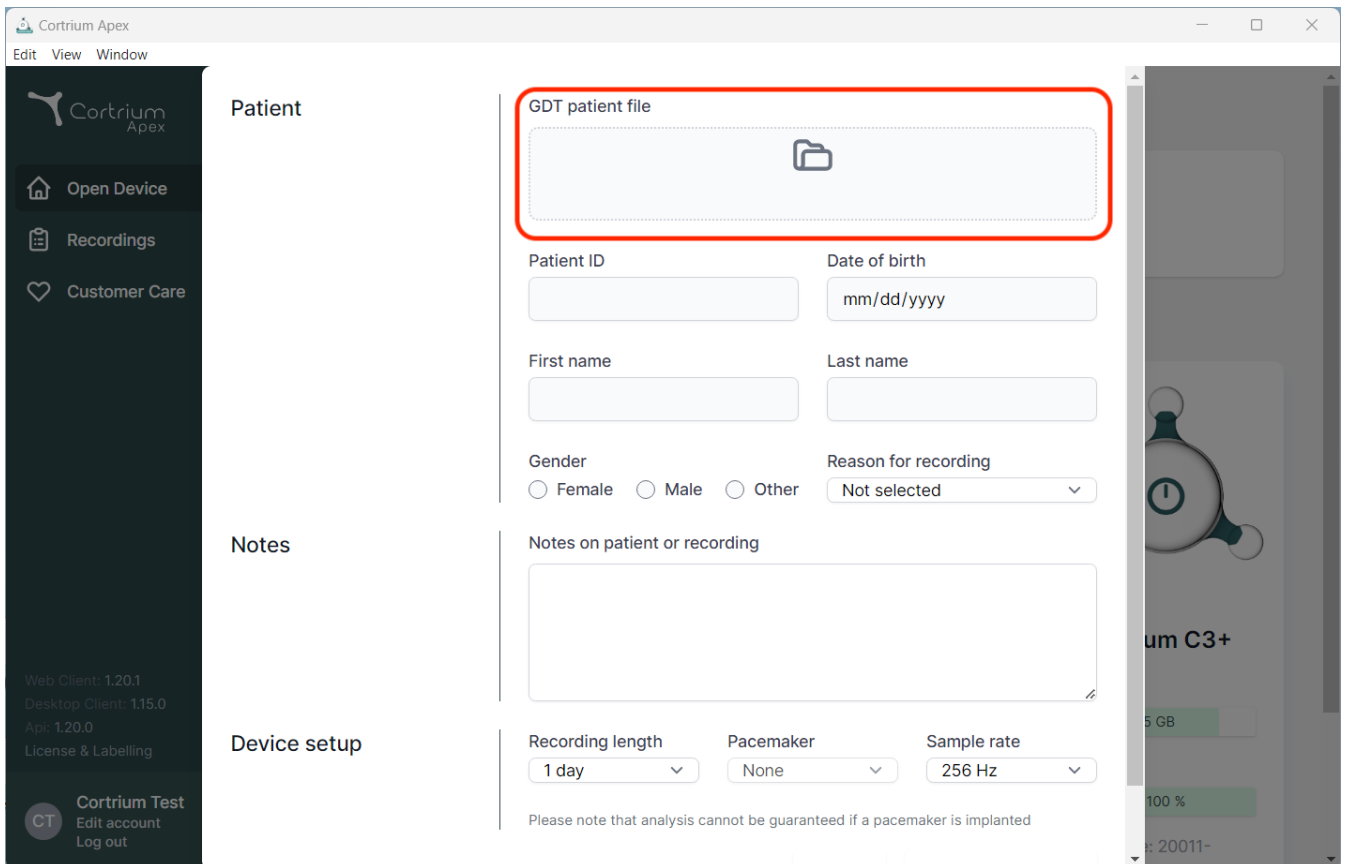
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections: 'Patient', 'Notes', and 'Device setup'. The 'Patient' section contains a 'GDT patient file' upload field (highlighted with a red box), followed by input fields for 'Patient ID', 'Date of birth' (with a 'mm/dd/yyyy' placeholder), 'First name', and 'Last name'. Below these are radio buttons for 'Gender' (Female, Male, Other) and a dropdown for 'Reason for recording' (currently 'Not selected'). The 'Notes' section has a text area for 'Notes on patient or recording'. The 'Device setup' section includes dropdowns for 'Recording length' (1 day), 'Pacemaker' (None), and 'Sample rate' (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. The right side of the interface shows a partial view of a device recording screen with a 'C3+' label and a 100% progress indicator.

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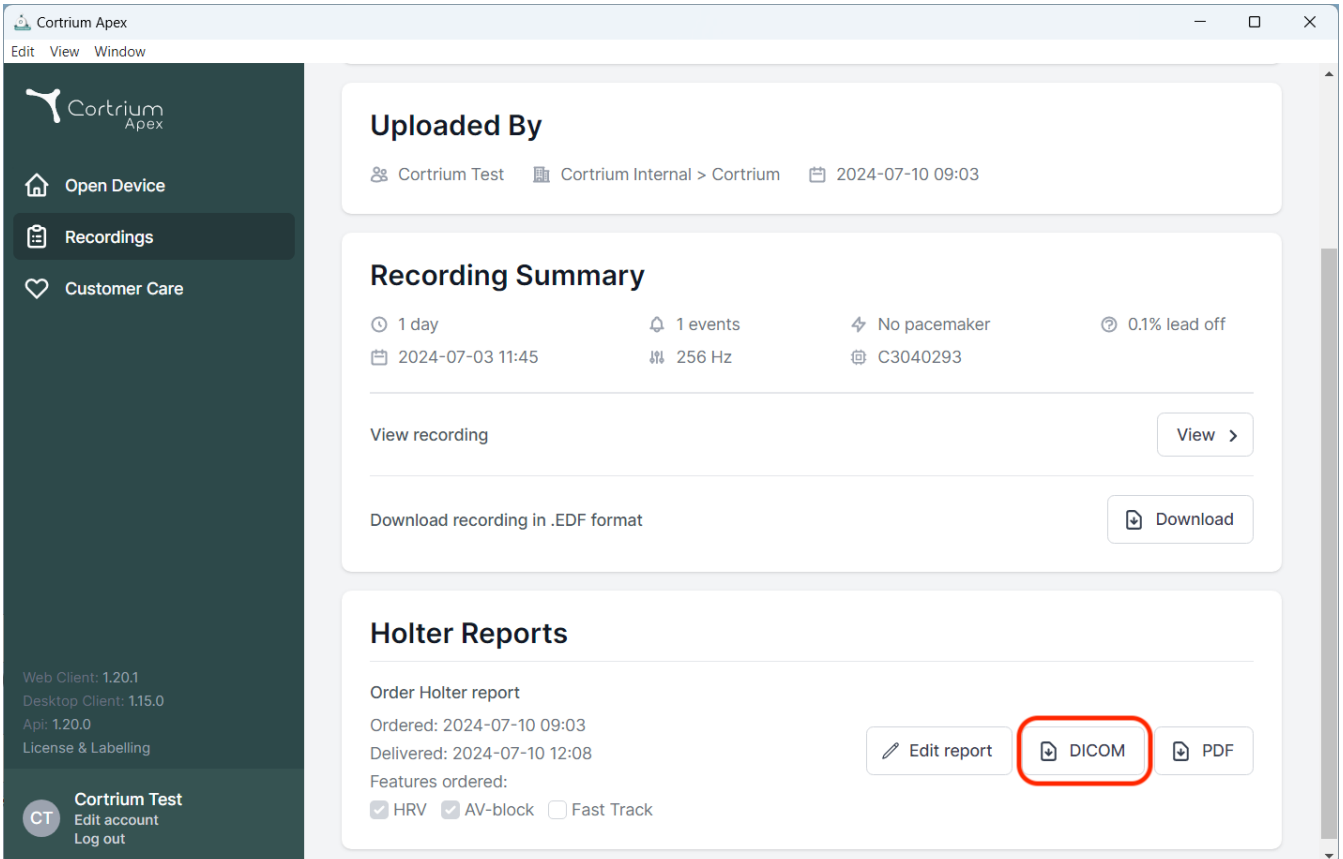The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:


- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' (2024-07-03 11:45), '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off' (C3040293). It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, the user 'Cortrium Test' is logged in, with options to 'Edit account' and 'Log out'.

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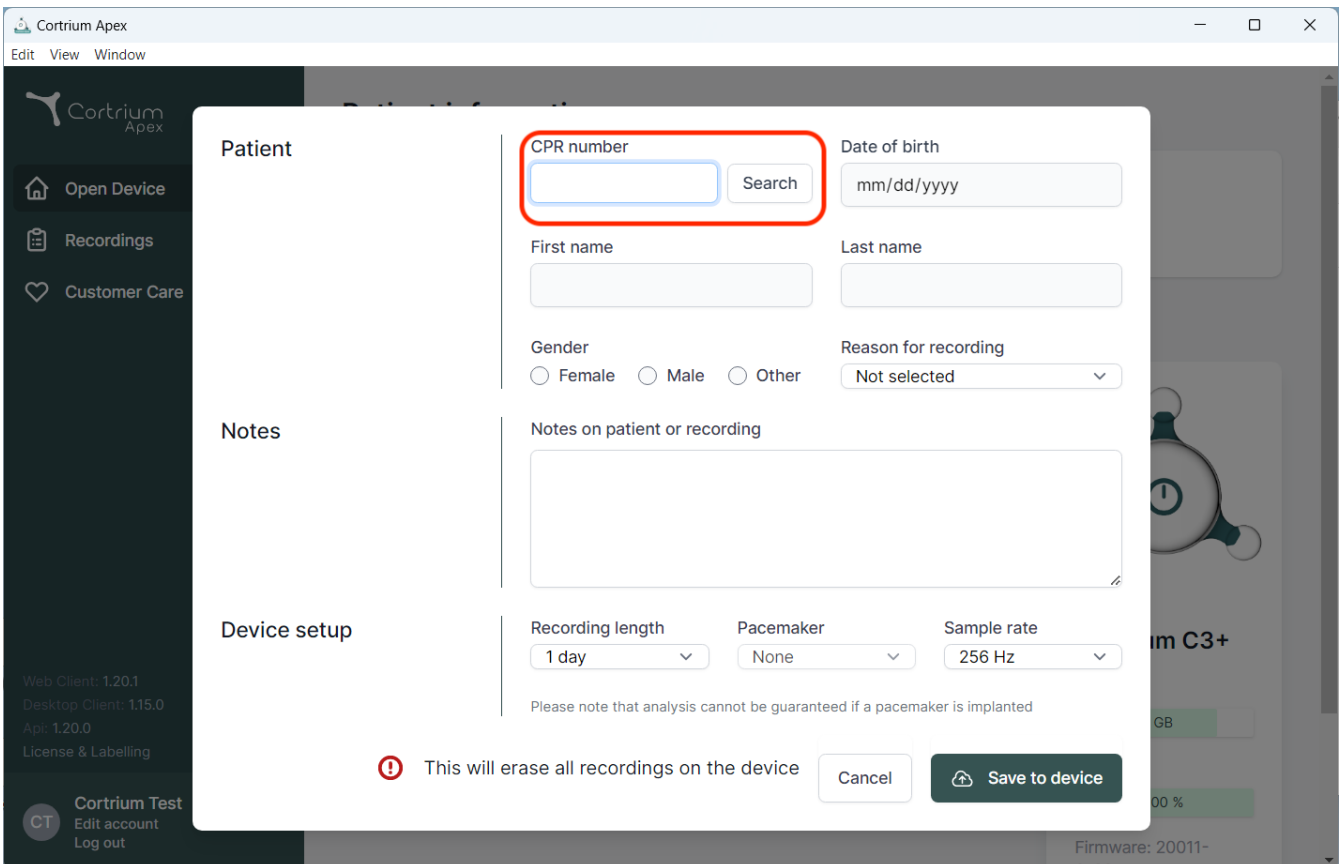Recordings

Customer Care

Web Client: 1.20.1  
Desktop Client: 1.15.0  
App: 1.20.0  
License & Labelling

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 This will erase all recordings on the device

Cancel

09/2024

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dänemark  
E-Mail: [info@cortrium.com](mailto:info@cortrium.com) • Website: [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033