# Cortrium Apex 1.20

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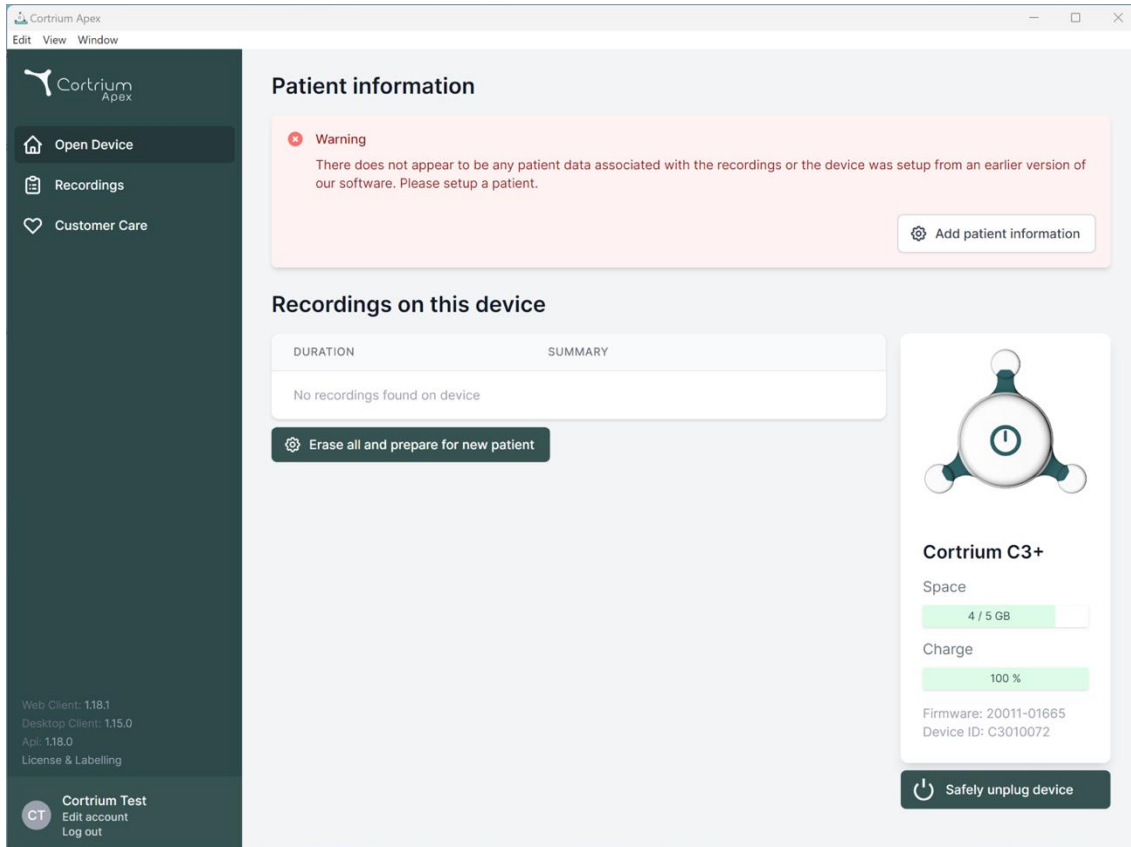
The screenshot shows the Cortrium Apex web interface. The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The main content area is titled "Patient information" and features a warning message: "Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient." Below this is a table for "Recordings on this device" with columns for DURATION and SUMMARY, showing "No recordings found on device". A button "Erase all and prepare for new patient" is visible. On the right, a device status card for "Cortrium C3+" displays "Space: 4 / 5 GB" and "Charge: 100 %". It also shows "Firmware: 20011-01665" and "Device ID: C3010072". A "Safely unplug device" button is at the bottom of the card. The bottom of the sidebar includes version information (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, License & Labelling) and user options (Cortrium Test, Edit account, Log out).

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Cortrium Apex  
Edit View Window

Cortrium Apex

Open Device  
Recordings  
Customer Care

### Patient information

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960  
Date of birth: 08/28/1960

First name: Cortrium  
Last name: Test

Gender:  Female  Male  Other  
Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day  
Pacemaker: None  
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

**⚠ This will erase all recordings on the device**     

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
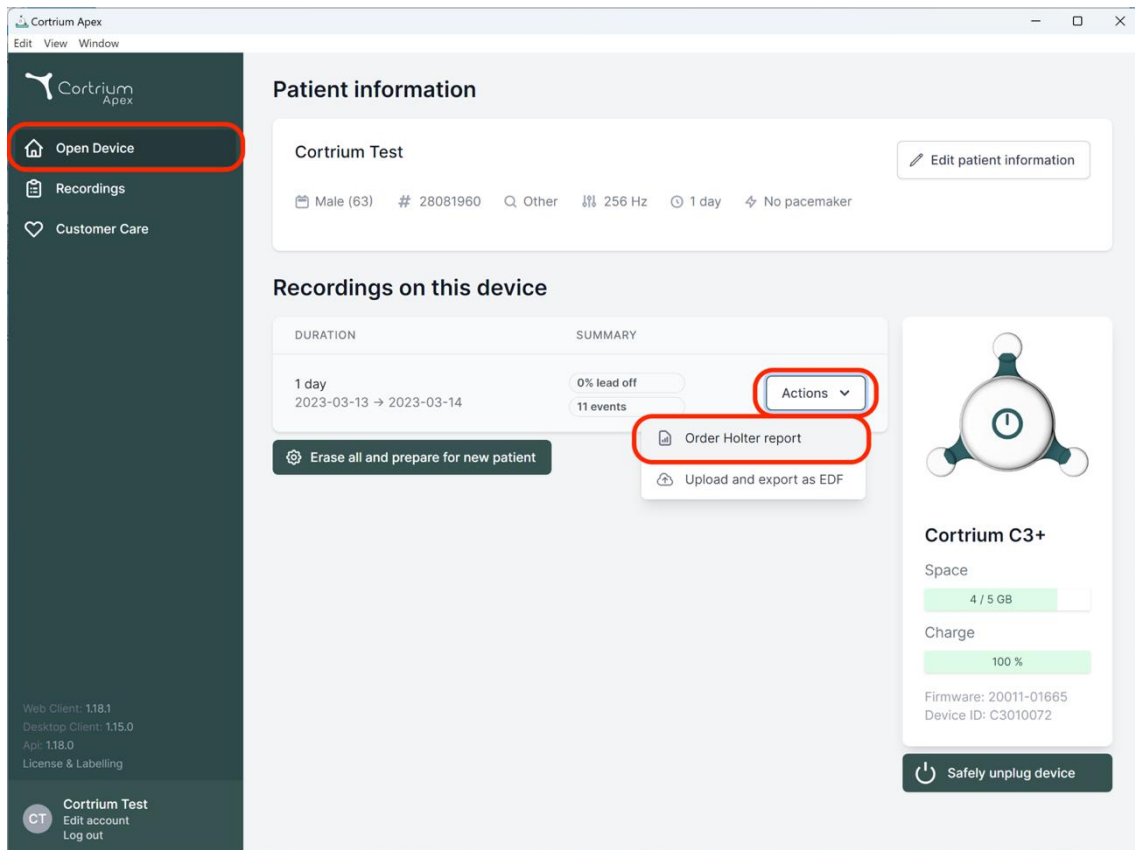
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device' (highlighted with a red circle), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including patient demographics and device settings. Below this is a 'Recordings on this device' section with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 11 events and 0% lead off. An 'Actions' dropdown menu is open, showing 'Order Holter report' (highlighted with a red circle) and 'Upload and export as EDF'. On the right, there is a device status card for 'Cortrium C3+' showing 4/5 GB space, 100% charge, and a 'Safely unplug device' button. The bottom of the sidebar shows the user's name 'Cortrium Test' and options to 'Edit account' or 'Log out'.

For at bestille Holter-rapporten skal du klikke på knappen " Handlinger" og derefter på "Bestil Holter-rapport".

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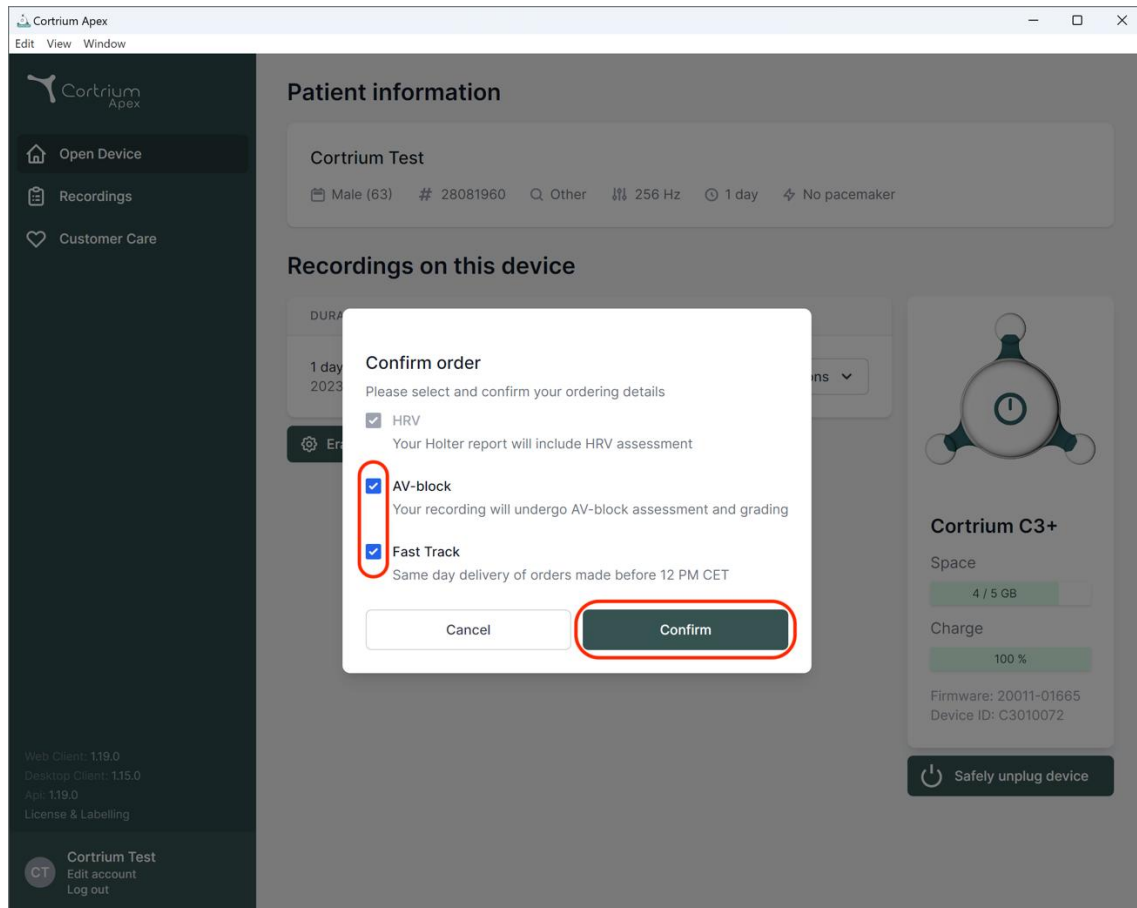
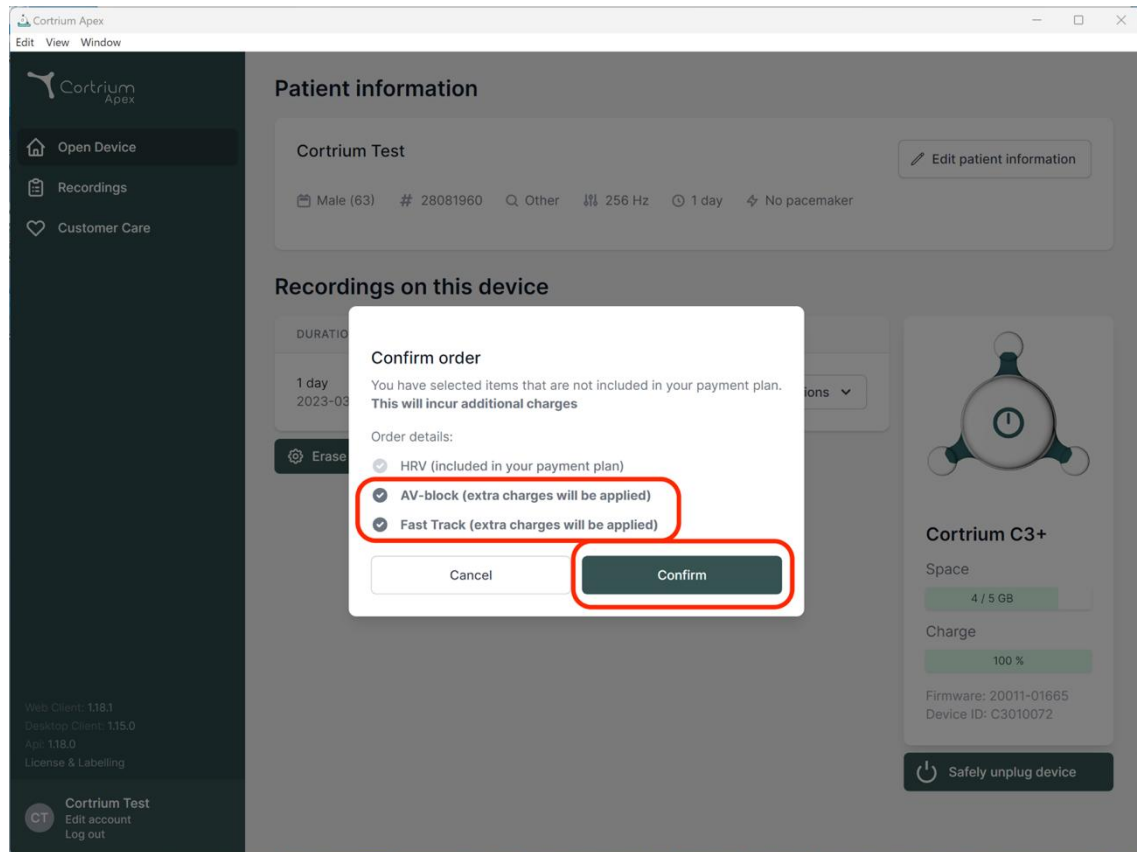
Herefter uploades optagelsen.

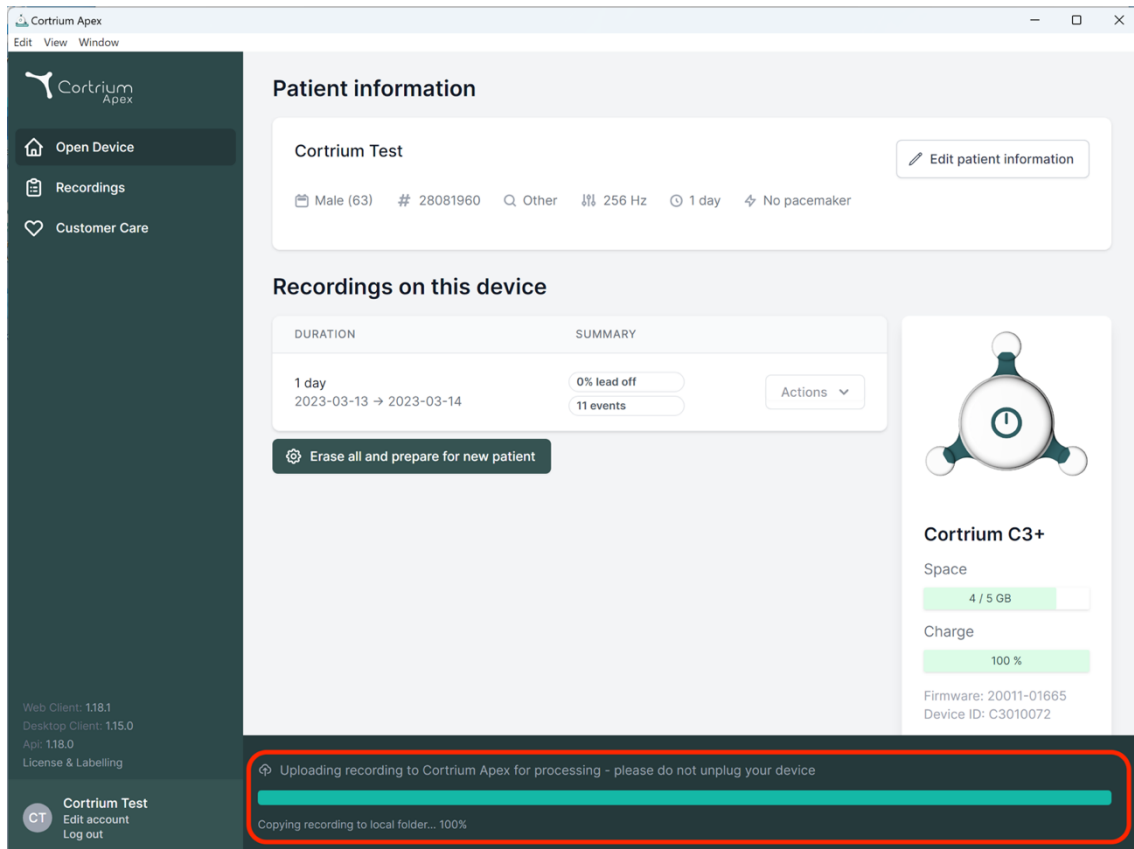The screenshot shows the Cortrium Apex web interface. The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and device settings (256 Hz, 1 day, No pacemaker). Below this is the 'Recordings on this device' section, showing a recording for '1 day' (2023-03-13 to 2023-03-14) with a '0% lead off' status and '11 events'. A 'Copy' icon is visible next to the recording duration. To the right, a 'Cortrium C3+' device status card shows 'Space' at 4/5 GB and 'Charge' at 100%. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. The bottom left corner shows the user's name 'Cortrium Test' and options to 'Edit account' and 'Log out'.

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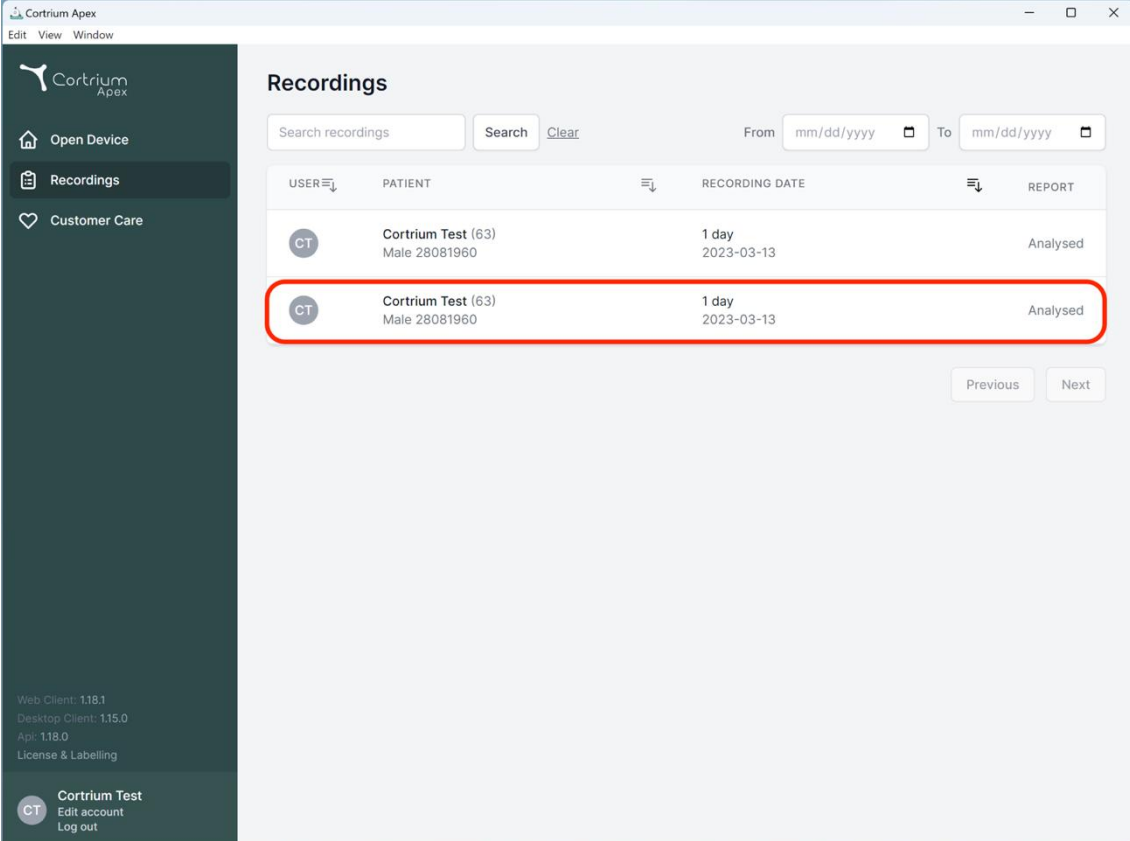
The screenshot shows the Cortrium Apex web application interface. The main content area is titled "Recordings" and contains a table with the following data:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

The second row of the table is highlighted with a red border. Below the table are "Previous" and "Next" navigation buttons. The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". At the bottom of the sidebar, the user is identified as "Cortrium Test" with options to "Edit account" and "Log out".

Cortrium Apex  
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1  
Desktop Client: 1.15.0  
App: 1.18.0  
License & Labelling

**CT** Cortrium Test  
Edit account  
Log out

### Recording details

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

#### Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

#### Recording Summary

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

#### Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
 HRV  AV-block  Fast Track

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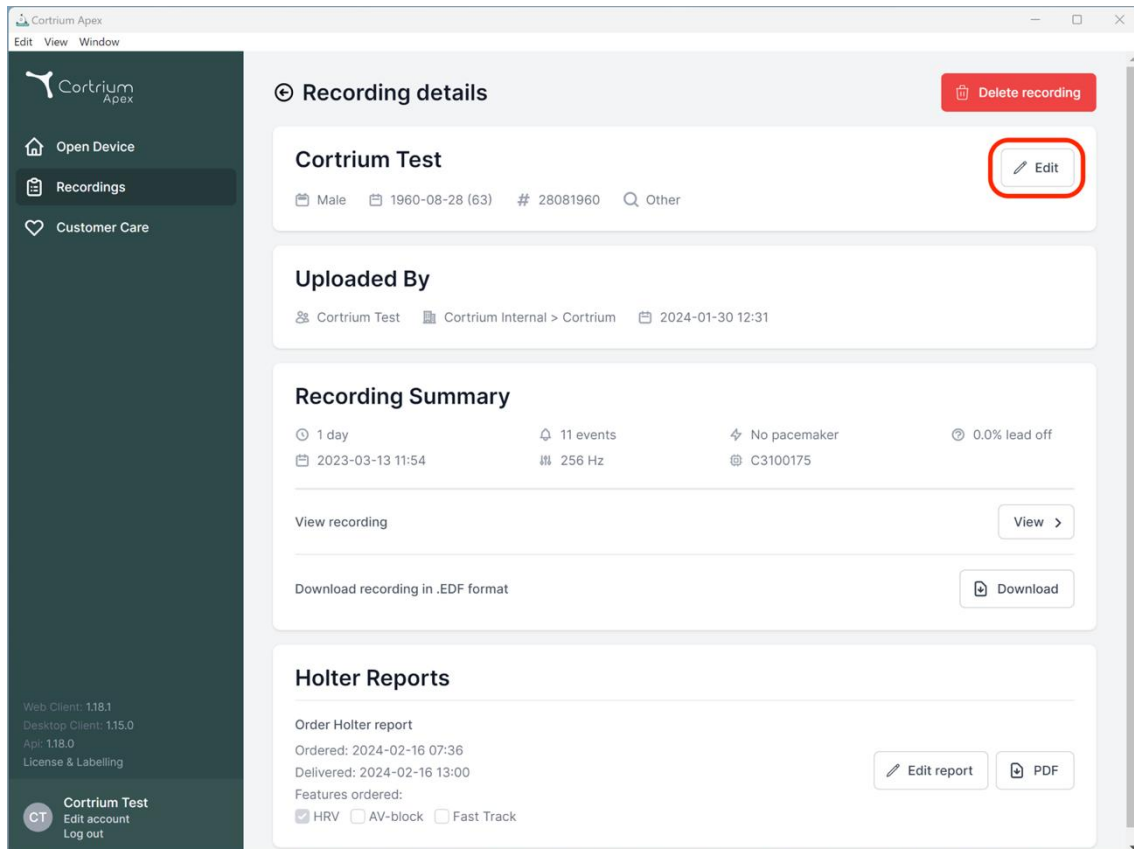
Patientoplysningerne kan redigeres ved at klikke på knappen "Rediger".

**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**  
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

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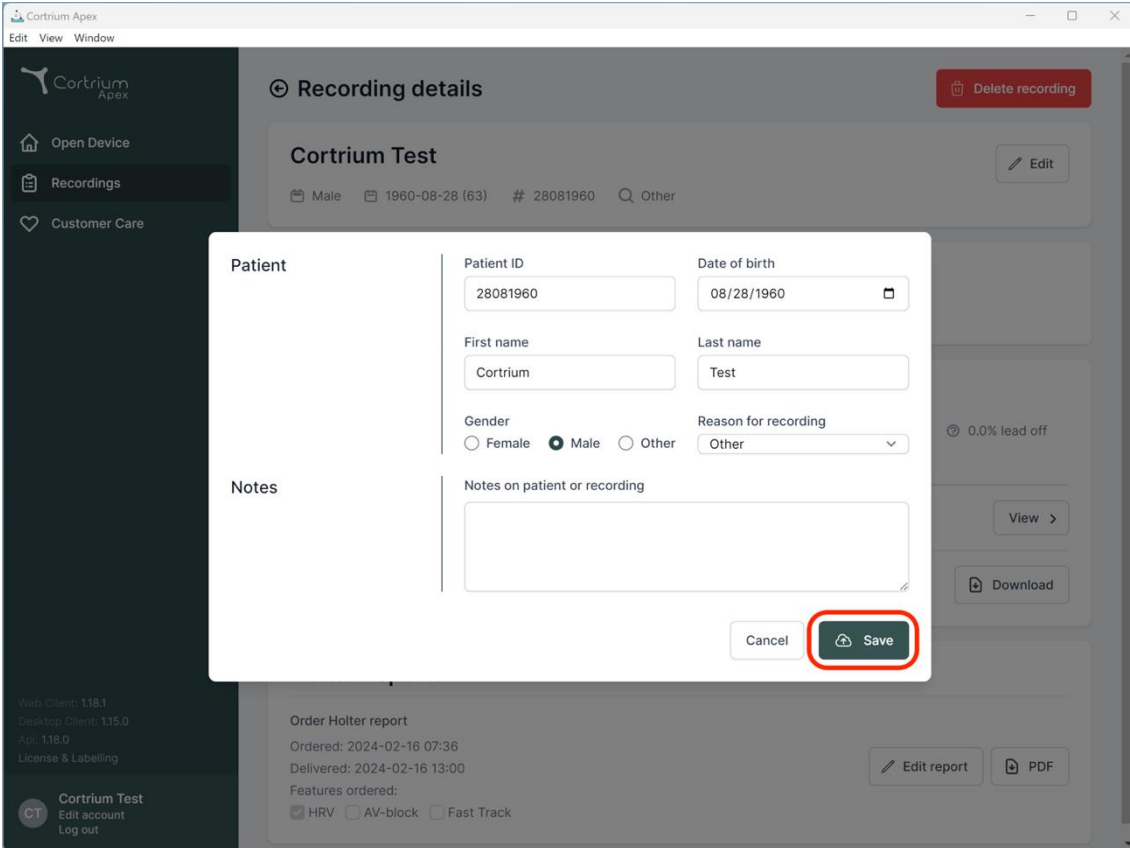
**Holter Reports**

Order Holter report  
Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
 HRV  AV-block  Fast Track
 Edit report PDF

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Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender:  Female  Male  Other Reason for recording: Other

**Notes**

Notes on patient or recording

Cancel Save

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

CT Cortrium Test  
Edit account  
Log out

Order Holter report  
Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00  
Features ordered:  
 HRV  AV-block  Fast Track

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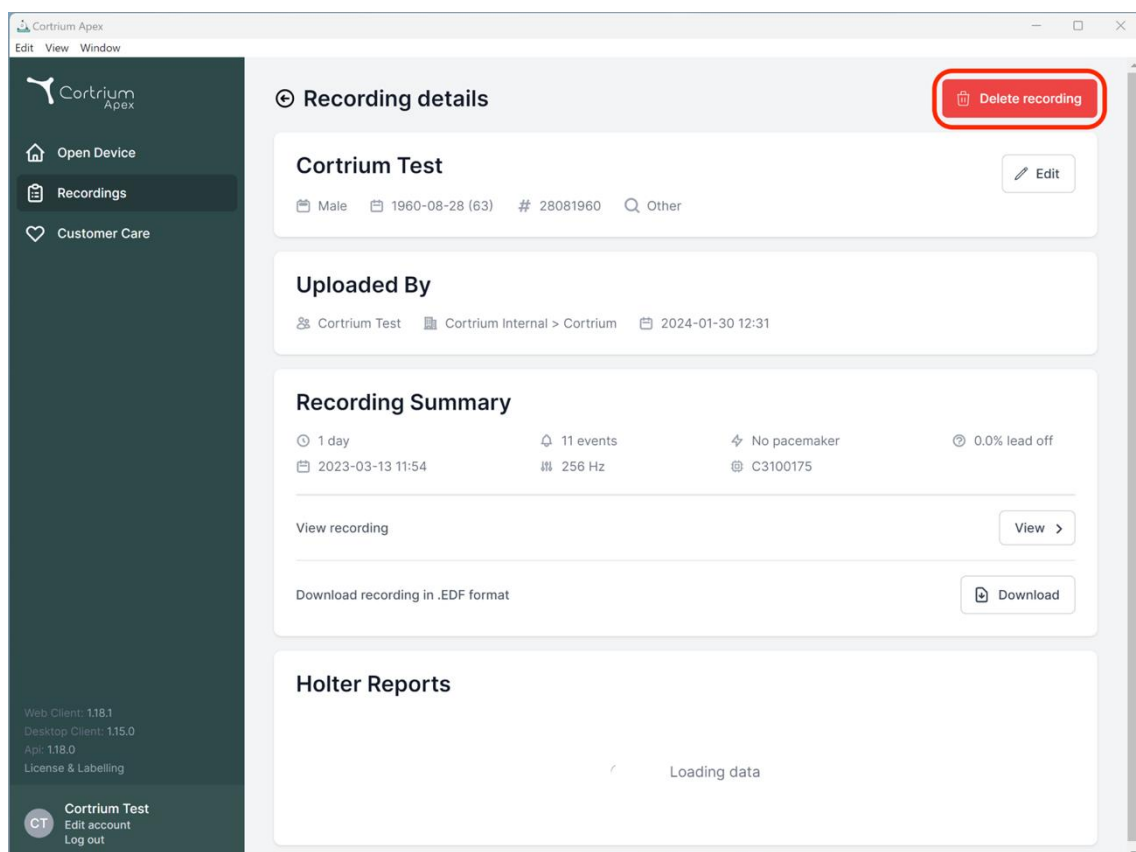
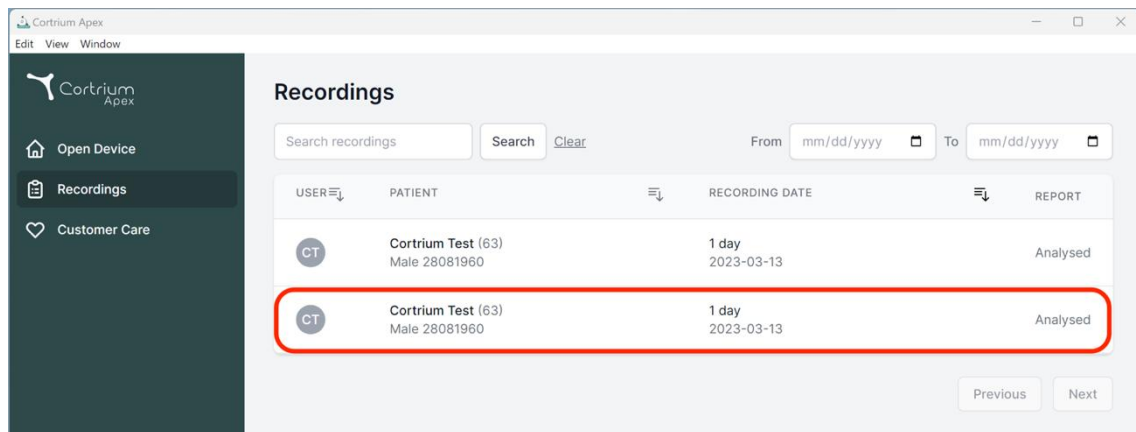
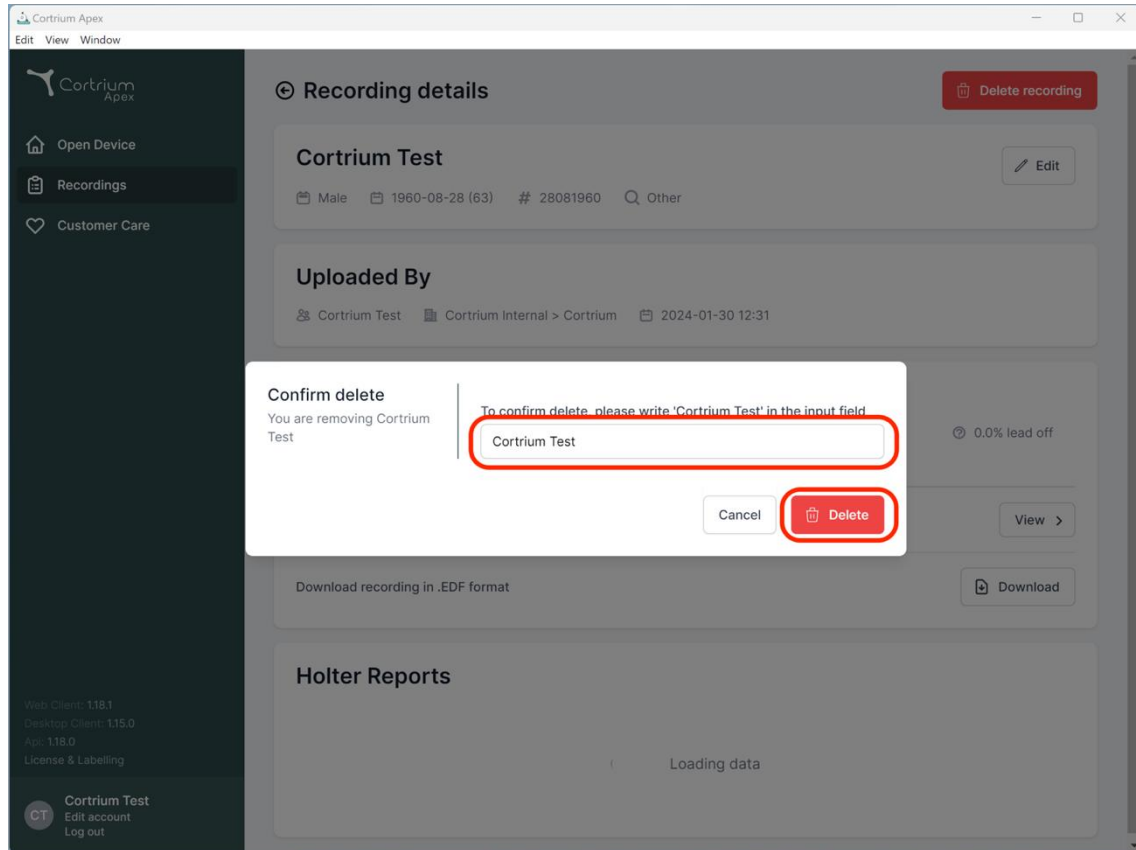
Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

CT Cortrium Test  
Edit account  
Log out

**Confirm delete**  
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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
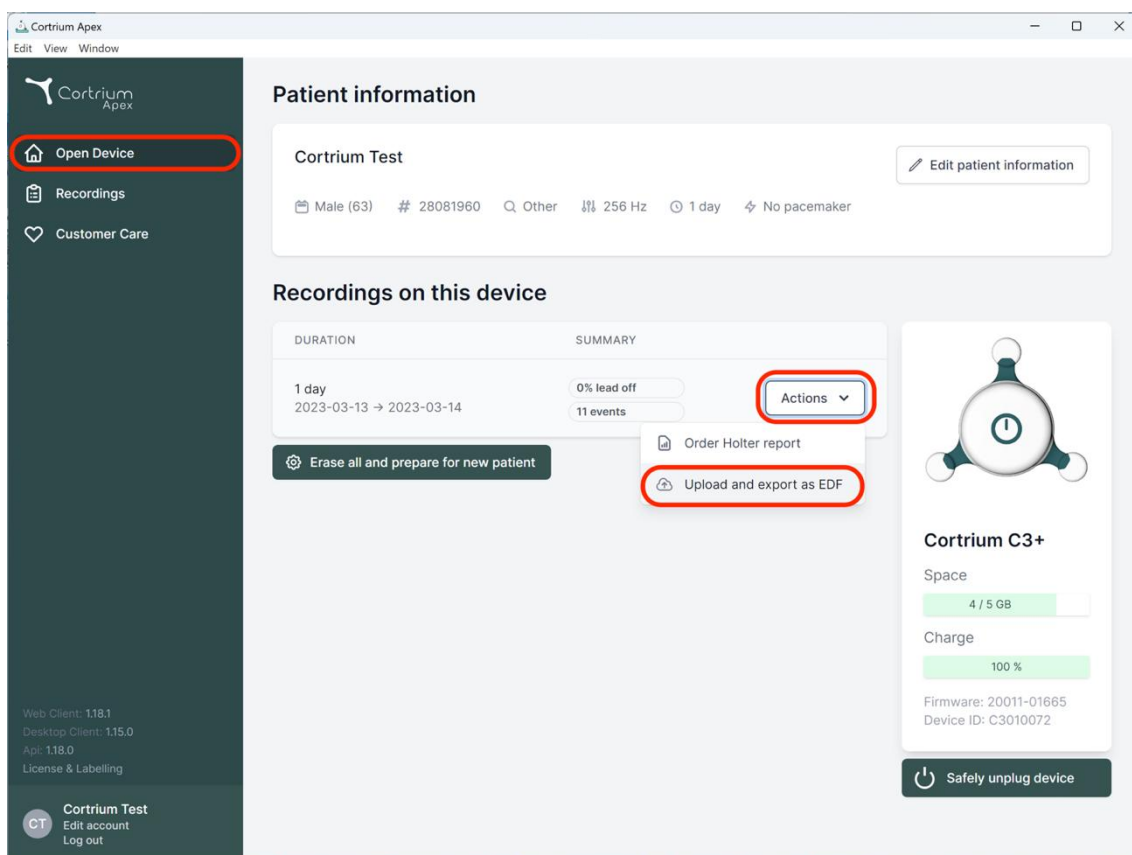
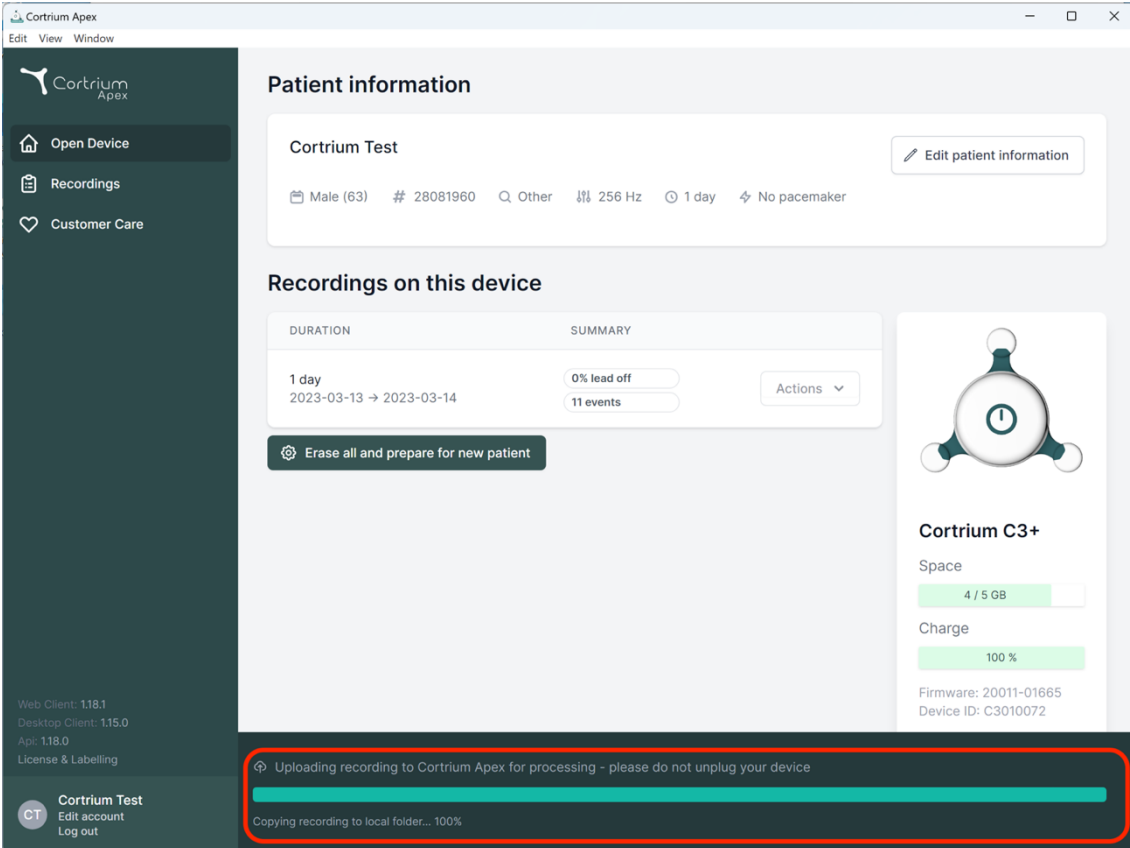
The screenshot shows the Cortrium Apex web interface. The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The main content area is titled "Patient information" and displays details for "Cortrium Test", including patient demographics and device settings. Below this, the "Recordings on this device" section shows a recording for "1 day" from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A red box highlights a progress bar at the bottom of the interface, indicating the upload of the recording to the Cortrium Apex server. The progress bar is labeled "Uploading recording to Cortrium Apex for processing - please do not unplug your device" and "Copying recording to local folder... 100%".

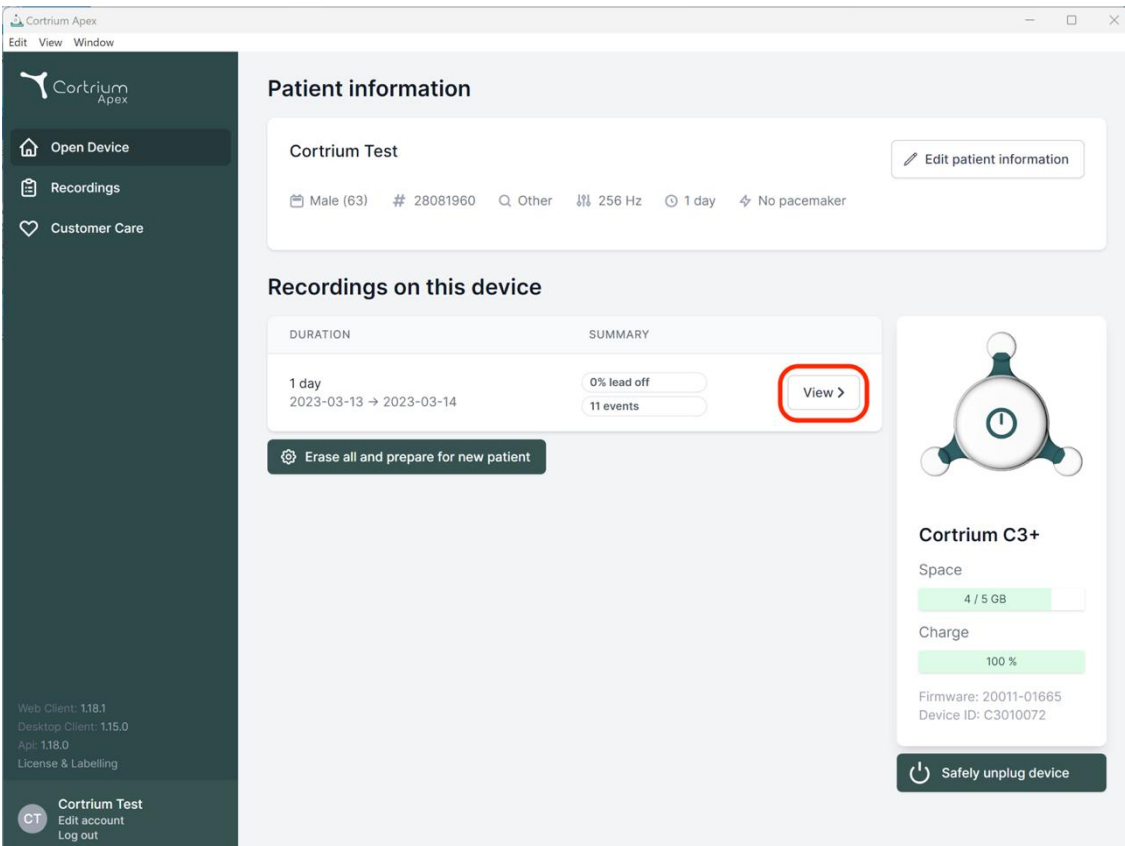The screenshot shows the Cortrium Apex web interface, similar to the previous one. The "Recordings on this device" section is highlighted, showing the recording details. A red box highlights the "View >" button next to the recording summary. The interface also displays the device status for "Cortrium C3+", including space usage (4 / 5 GB) and charge level (100%). A "Safely unplug device" button is visible at the bottom right of the device status panel.

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
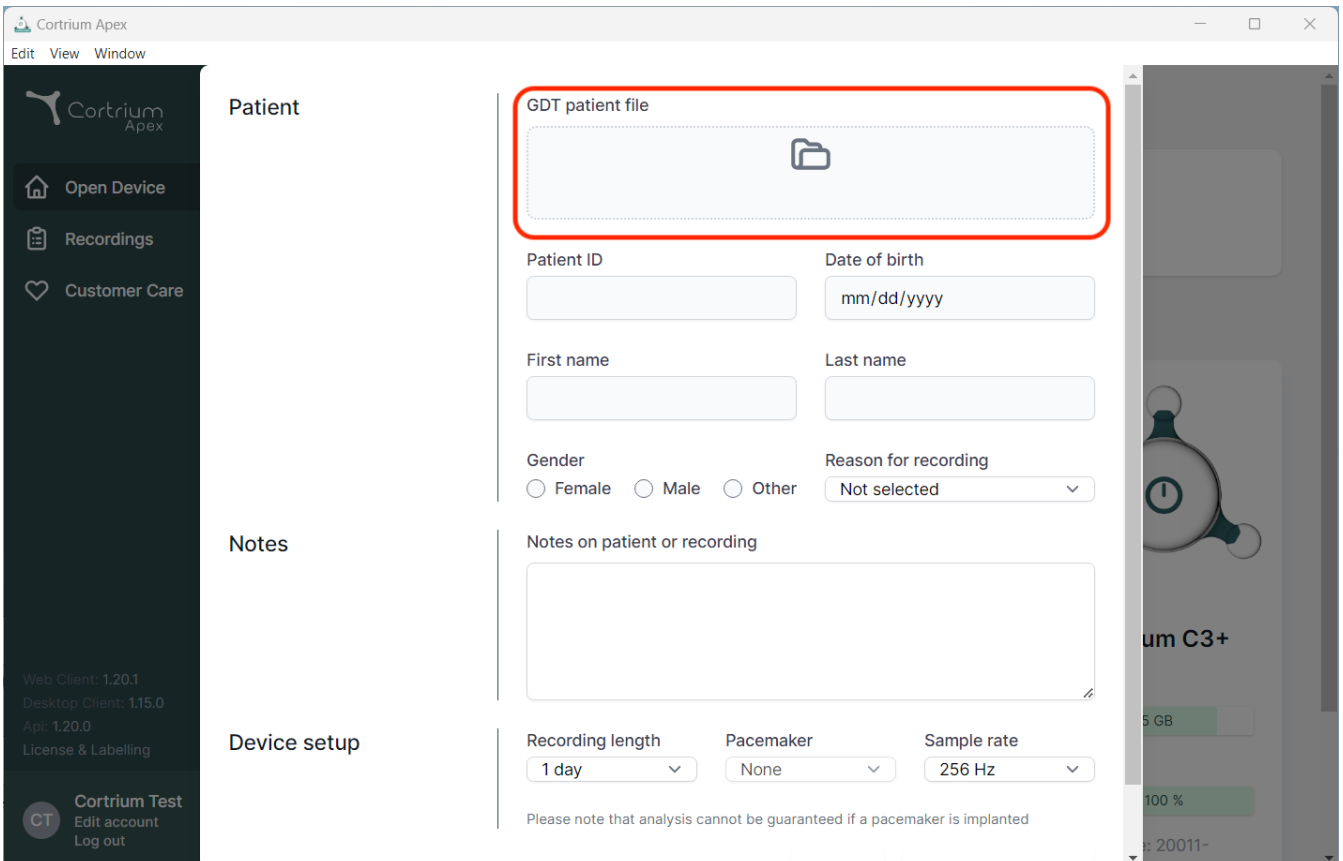
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red box highlights the "GDT patient file" field, which is currently empty and contains a folder icon. Below this field are several input fields: "Patient ID", "Date of birth" (with a placeholder "mm/dd/yyyy"), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected"). Below the patient information fields is a "Notes" section with a text area labeled "Notes on patient or recording". At the bottom of the form is the "Device setup" section, which includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below the device setup section states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar shows the user's name "Cortrium Test" and options to "Edit account" and "Log out". The top of the browser window shows the title "Cortrium Apex" and standard window controls.

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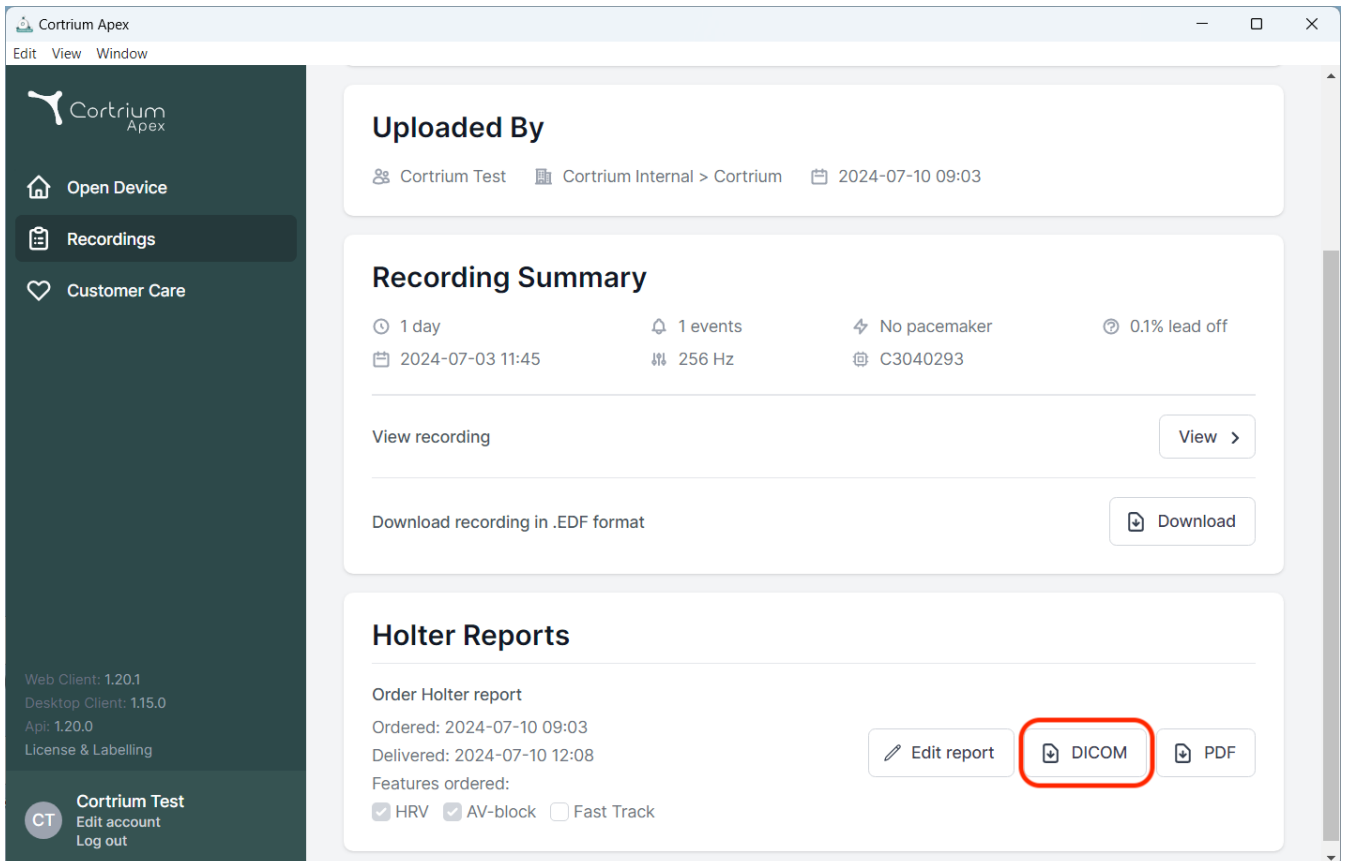The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections: 'Uploaded By', 'Recording Summary', and 'Holter Reports'. The 'Holter Reports' section shows the report order details, including the date ordered (2024-07-10 09:03) and delivered (2024-07-10 12:08). Below this, there are three buttons: 'Edit report', 'DICOM', and 'PDF'. The 'DICOM' button is highlighted with a red rectangular box, indicating the action to be taken to download the DICOM report.

**Uploaded By**  
Cortrium Test | Cortrium Internal > Cortrium | 2024-07-10 09:03

**Recording Summary**  
1 day | 1 events | No pacemaker | 0.1% lead off  
2024-07-03 11:45 | 256 Hz | C3040293

View recording [View >](#)

Download recording in .EDF format [Download](#)

**Holter Reports**

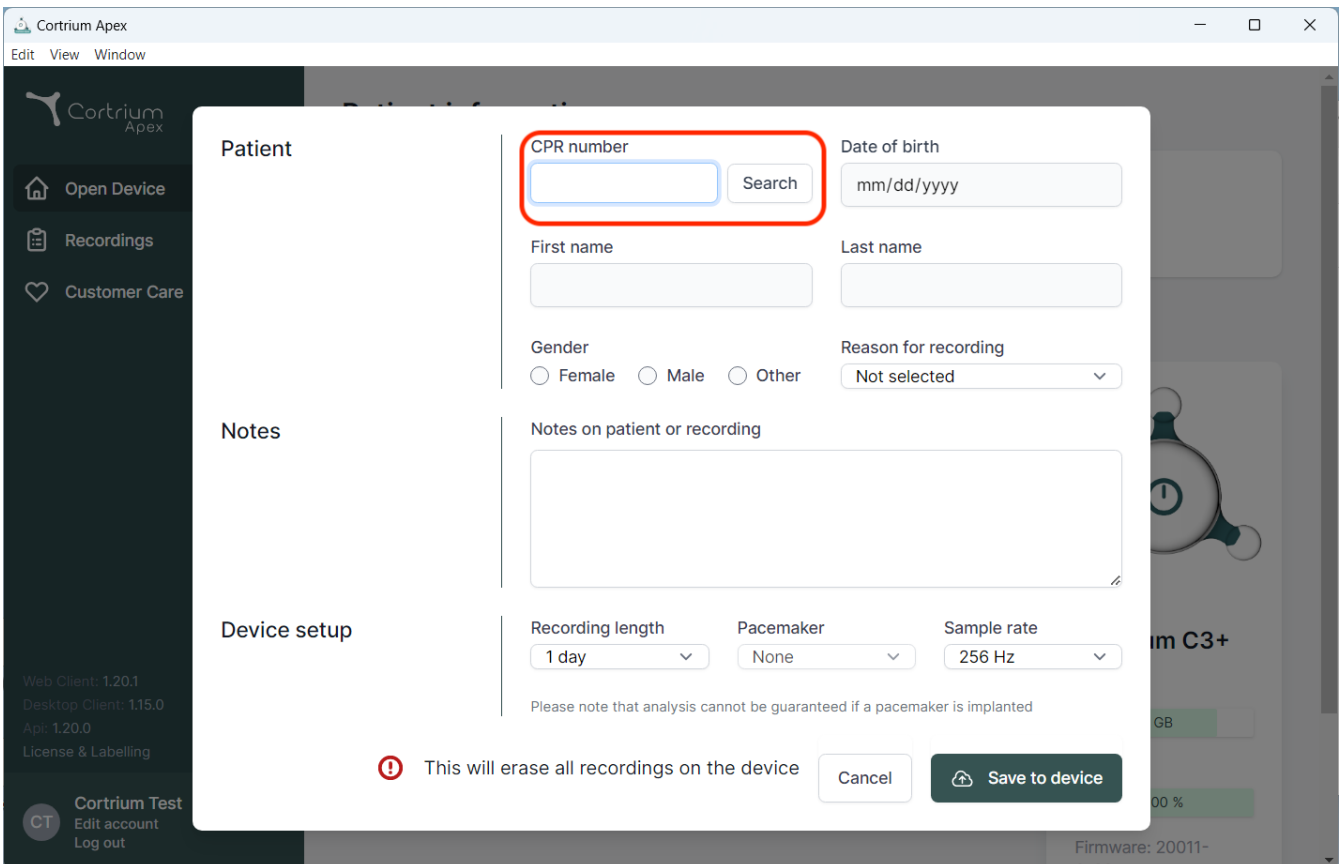
Order Holter report  
Ordered: 2024-07-10 09:03  
Delivered: 2024-07-10 12:08  
Features ordered:  
 HRV  AV-block  Fast Track

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The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, displaying a search form. The "CPR number" field is highlighted with a red box. The form includes the following fields and options:

- CPR number:** A text input field with a "Search" button next to it.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the form, a warning message is displayed: "This will erase all recordings on the device". At the bottom right of the modal, there are "Cancel" and "Save to device" buttons.

09/2024

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark  
E-mail: [info@cortrium.com](mailto:info@cortrium.com) • Hjemmeside: [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033