# Cortrium Apex 1.20

## Kasutusjuhend

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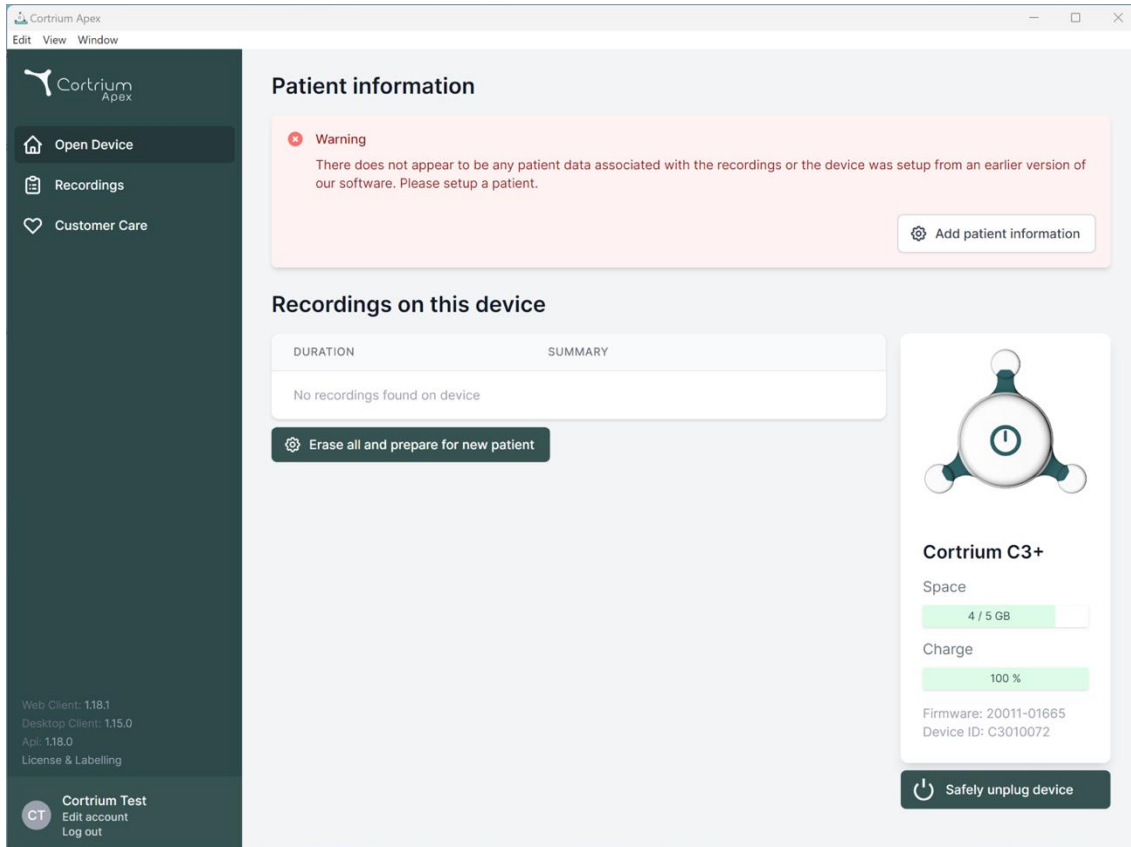
The screenshot shows the Cortrium Apex web interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. Below these are version numbers (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0) and a user profile for 'Cortrium Test' with 'Edit account' and 'Log out' options. The main content area is titled 'Patient information' and features a warning message: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table header for 'DURATION' and 'SUMMARY', showing 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' displays 'Space: 4 / 5 GB' and 'Charge: 100 %', along with 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex  
Edit View Window

Cortrium Apex

Open Device  
Recordings  
Customer Care

### Patient information

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960  
Date of birth: 08/28/1960

First name: Cortrium  
Last name: Test

Gender:  Female  Male  Other  
Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day  
Pacemaker: None  
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

**⚠ This will erase all recordings on the device**   Cancel   **Save to device**

Cortrium C3+  
pace  
4 / 5 GB  
charge  
100 %  
firmware: 20011-01665  
device ID: C3010072

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
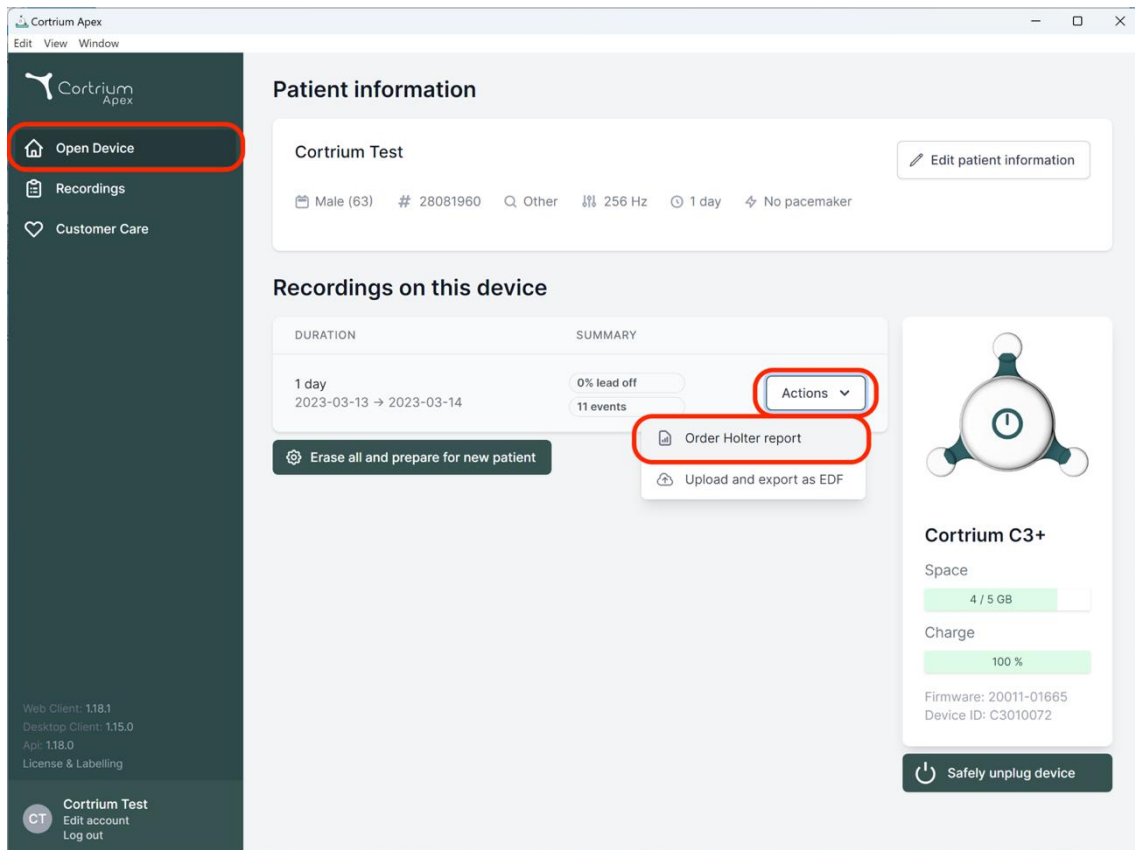
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device' (highlighted with a red box), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, the 'Recordings on this device' section shows a recording with 11 events and a '0% lead off' status. An 'Actions' dropdown menu is open, with 'Order Holter report' (highlighted with a red box) and 'Upload and export as EDF' visible. On the right, a device status card for 'Cortrium C3+' shows 4/5 GB space, 100% charge, and a 'Safely unplug device' button. The bottom left of the sidebar shows the user's name 'Cortrium Test' and options to 'Edit account' or 'Log out'.

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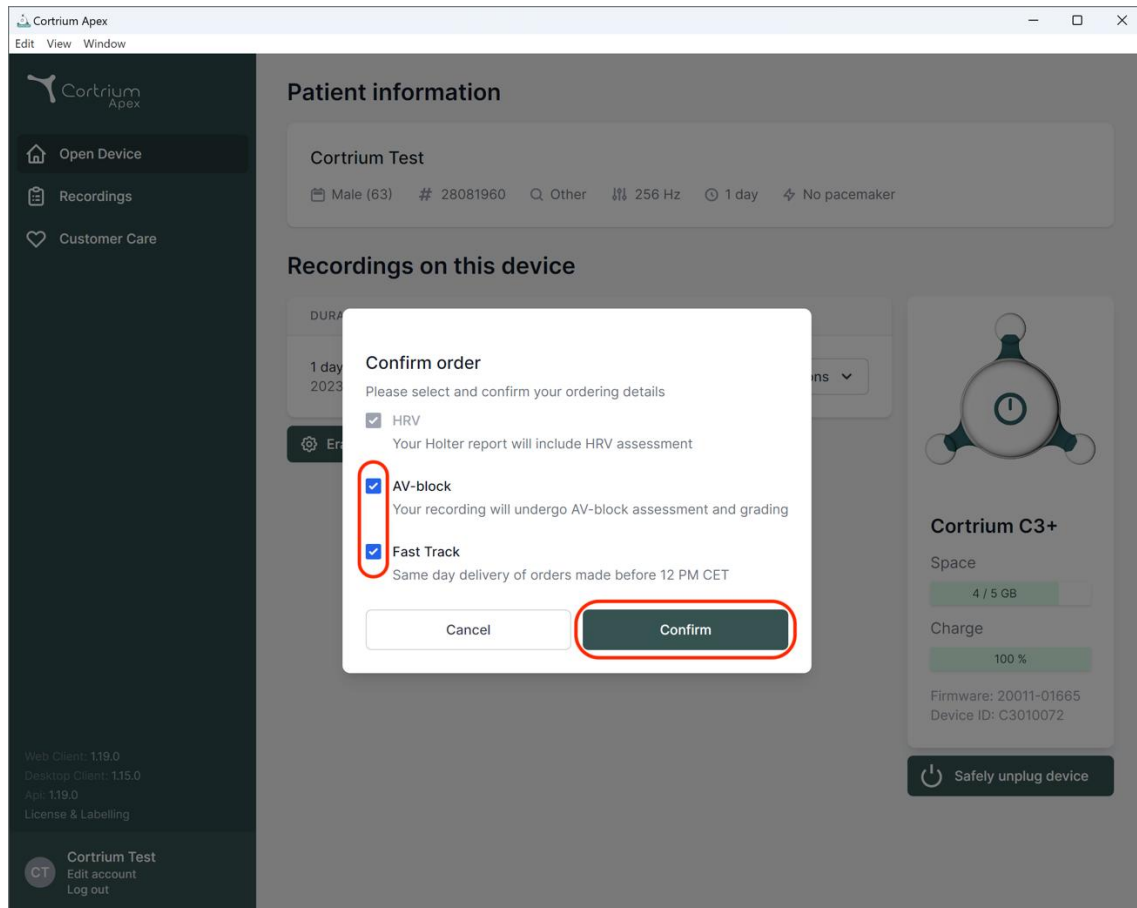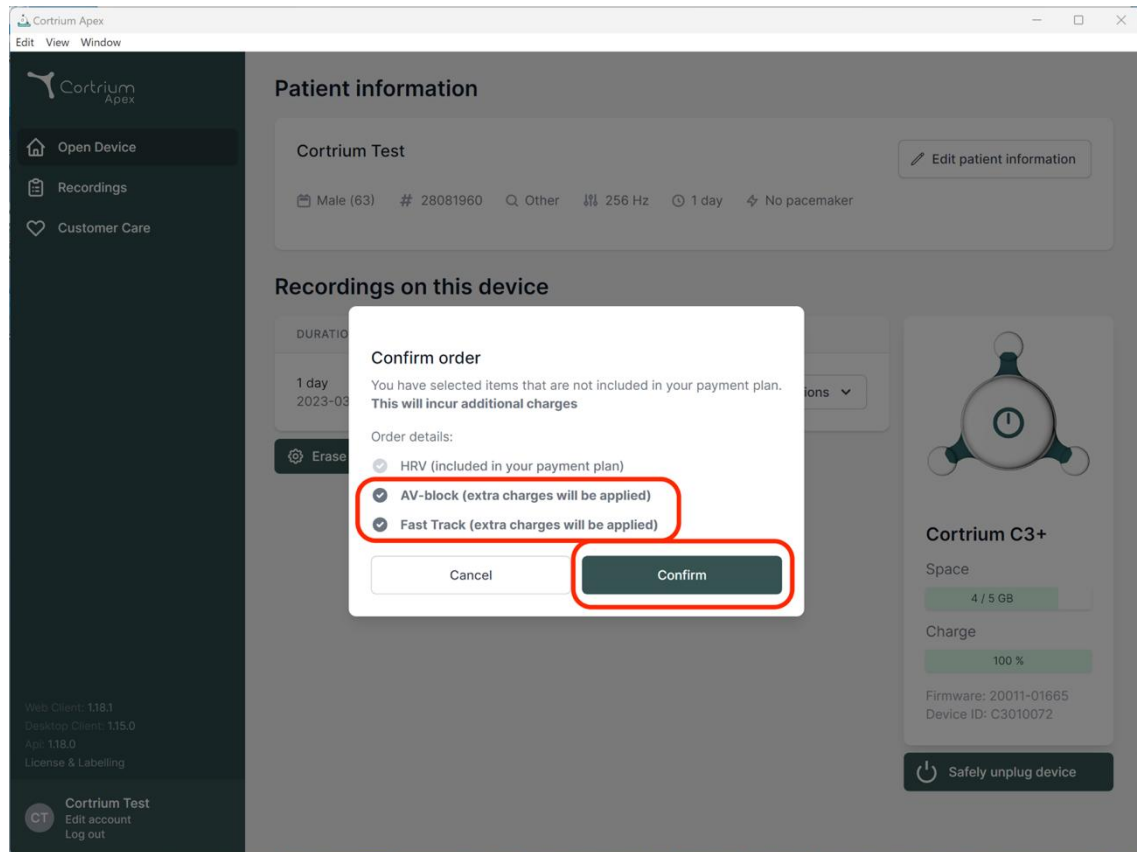
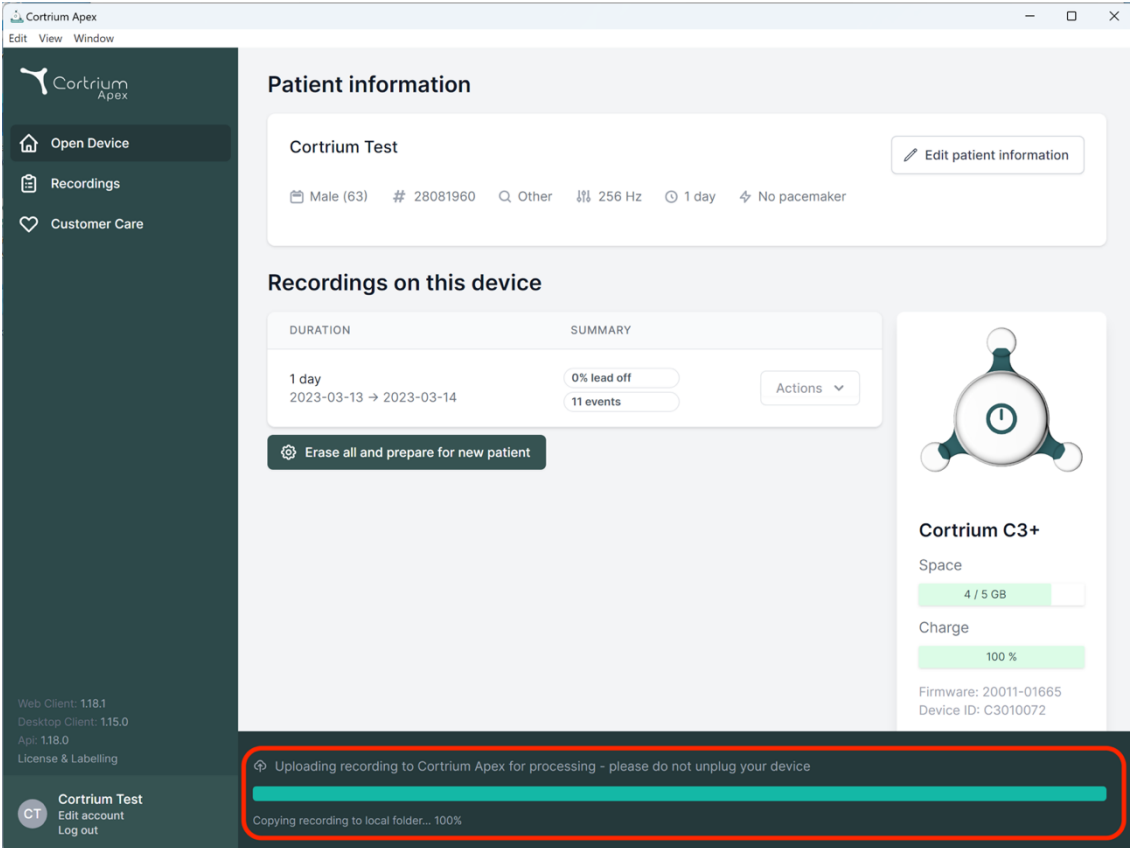
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient information" and shows details for "Cortrium Test", including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). It also indicates 0% lead off and 11 events. A sidebar on the left contains navigation options: "Open Device", "Recordings", and "Customer Care". Below the patient information, there is a section for "Recordings on this device" with a table showing the duration and summary. A "Erase all and prepare for new patient" button is visible. On the right, there is a device status card for "Cortrium C3+" showing 4/5 GB space and 100% charge. At the bottom, a red-bordered box highlights a progress bar and status message: "Uploading recording to Cortrium Apex for processing - please do not unplug your device" and "Copying recording to local folder... 100%".

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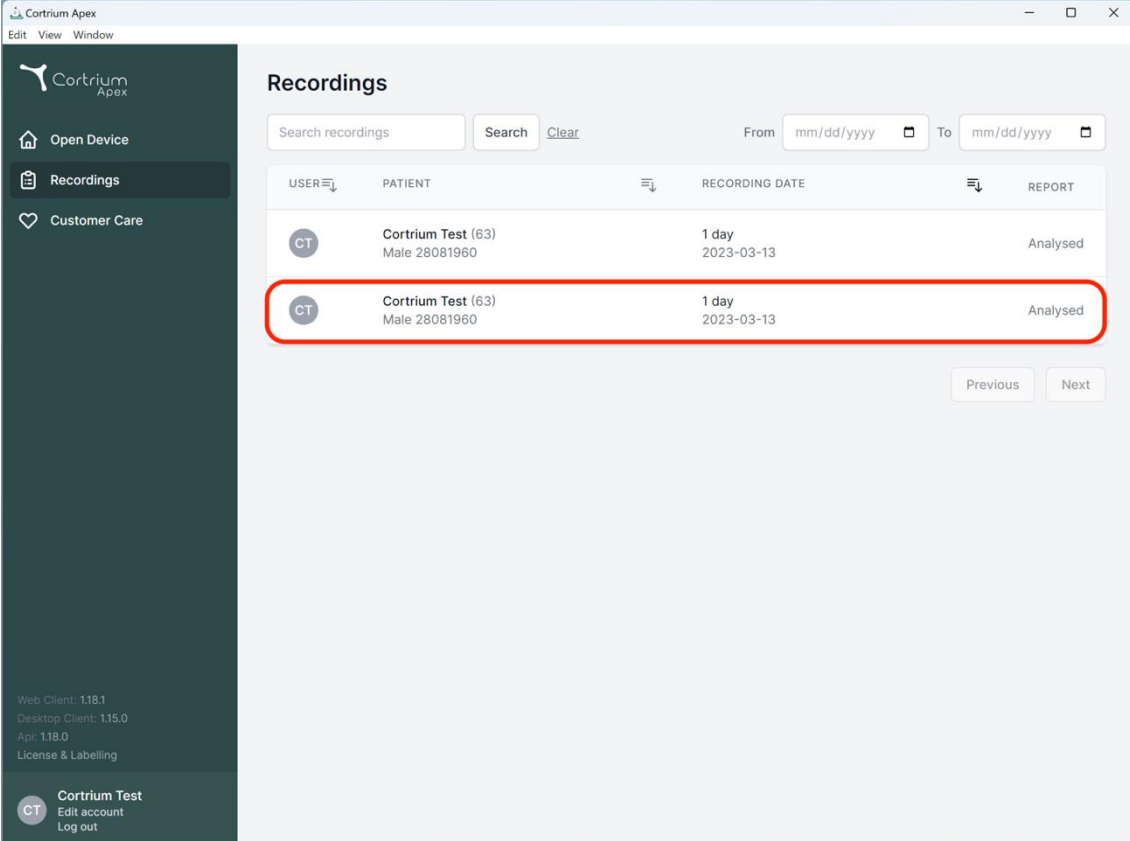
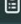
The screenshot shows the Cortrium Apex web application interface. The main content area is titled "Recordings" and contains a table with the following columns: USER, PATIENT, RECORDING DATE, and REPORT. The table has two rows of data, both of which are highlighted with a red border. The first row shows a recording by user "CT" for patient "Cortrium Test (63) Male 28081960" on "2023-03-13" with a "1 day" duration and "Analysed" status. The second row is identical. Below the table are "Previous" and "Next" navigation buttons. The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". At the bottom of the sidebar, there is a user profile for "Cortrium Test (63)" with options to "Edit account" and "Log out".

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

Cortrium Apex  
Edit View Window

 Cortrium Apex

 Open Device

 Recordings

 Customer Care

Web Client: 1.18.1  
Desktop Client: 1.15.0  
App: 1.18.0  
License & Labelling

 **Cortrium Test**  
Edit account  
Log out

### Recording details

 Delete recording

#### Cortrium Test

 Male  1960-08-28 (63)  # 28081960  Other  Edit

#### Uploaded By

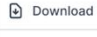
 Cortrium Test  Cortrium Internal > Cortrium  2024-01-30 12:31

#### Recording Summary

 1 day  11 events  No pacemaker  0.0% lead off

 2023-03-13 11:54  256 Hz  C3100175

View recording  View >

Download recording in .EDF format  Download

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
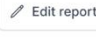
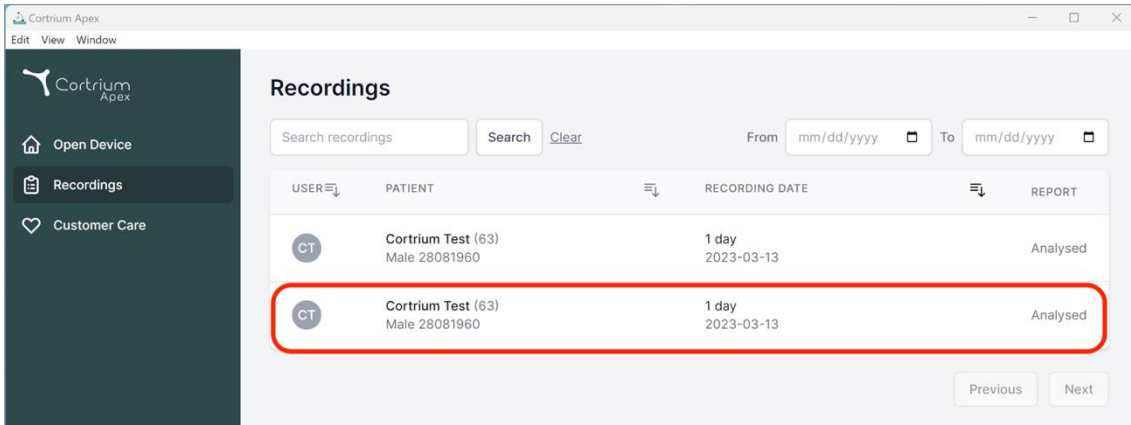
Recordings

Search recordings Search Clear From mm/dd/yyyy To mm/dd/yyyy

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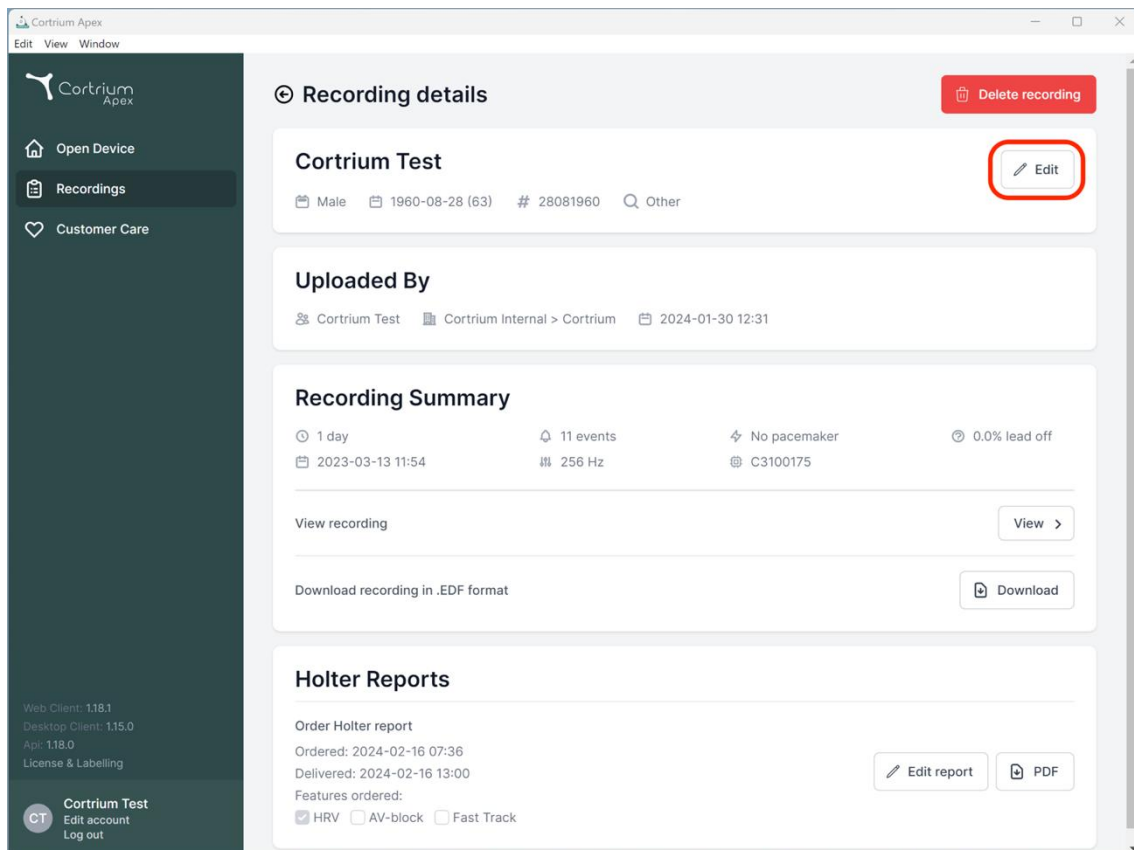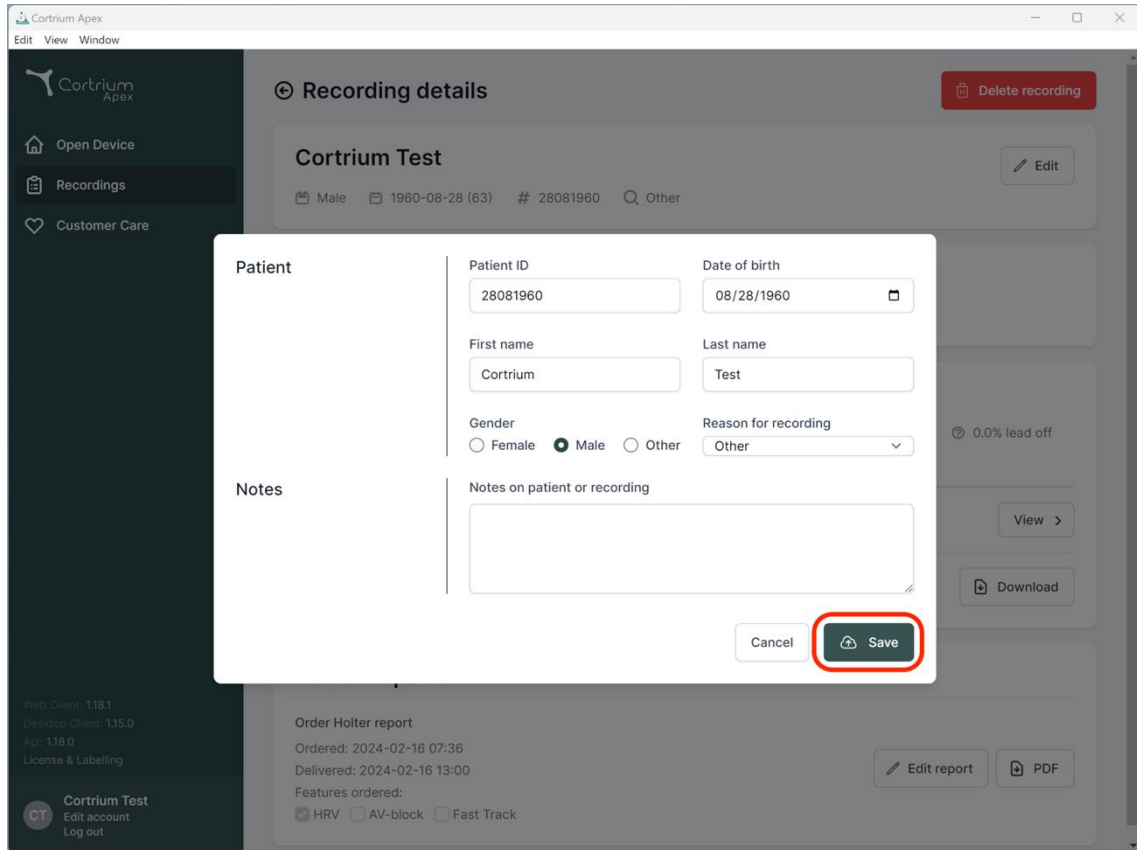
Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender:  Female  Male  Other Reason for recording: Other

**Notes**

Notes on patient or recording

Cancel Save

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

CT Cortrium Test  
Edit account  
Log out

Order Holter report  
Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00  
Features ordered:  
 HRV  AV-block  Fast Track

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
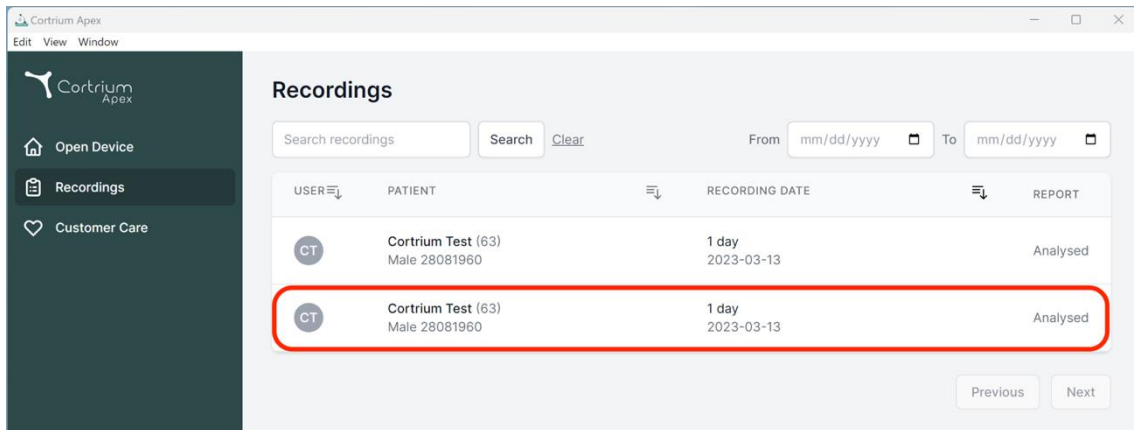
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. A table lists recordings with columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row is highlighted with a red border.

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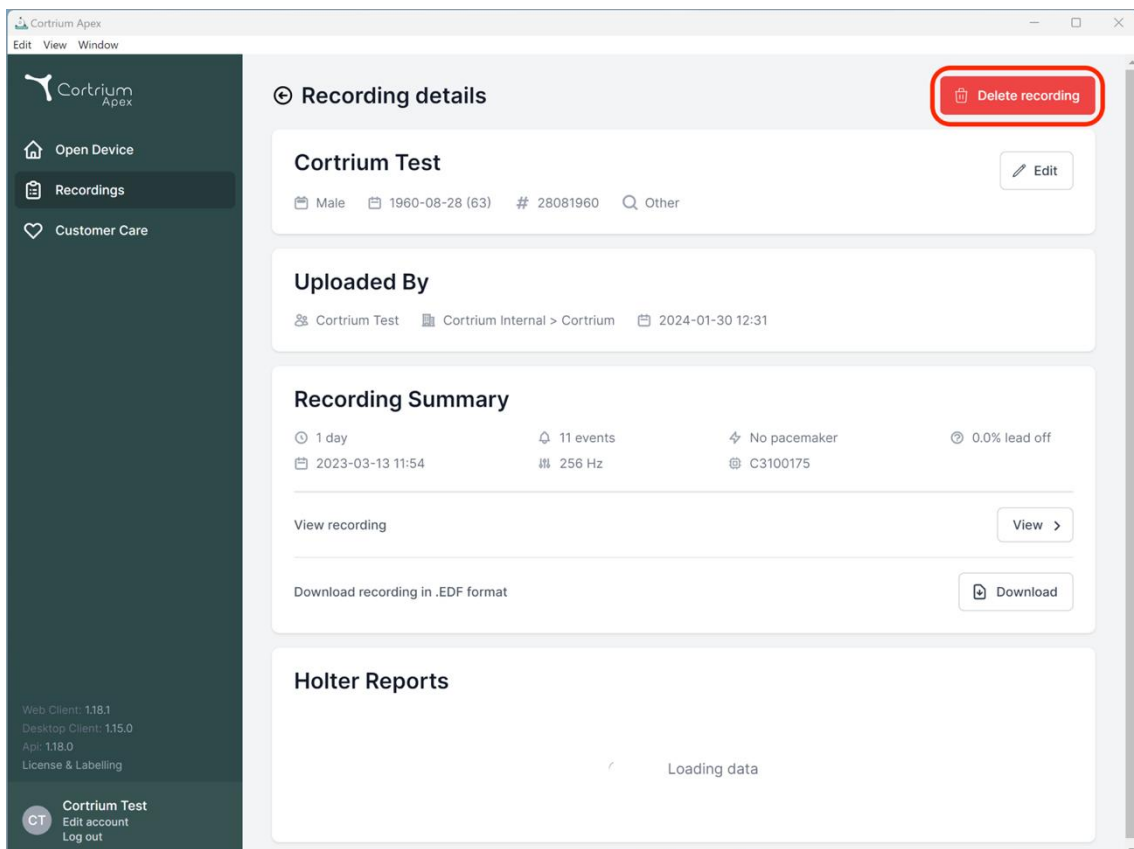The screenshot shows the 'Recording details' page for a specific recording. A red box highlights the 'Delete recording' button in the top right corner.

**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

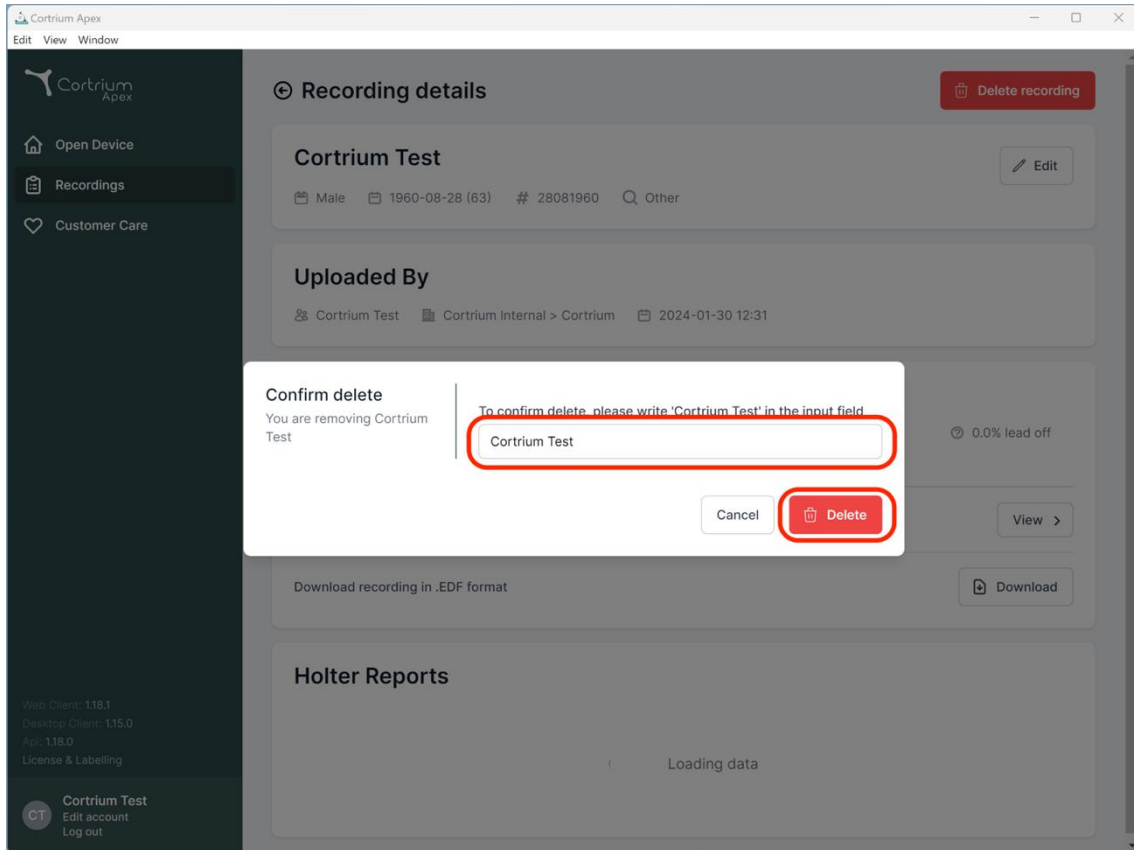
1 day 11 events No pacemaker 0.0% lead off  
 2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

**Holter Reports**

Loading data

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

### Recording details

Delete recording

#### Cortrium Test

Male 1960-08-28 (63) # 28081960 Other

Edit

#### Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

#### Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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
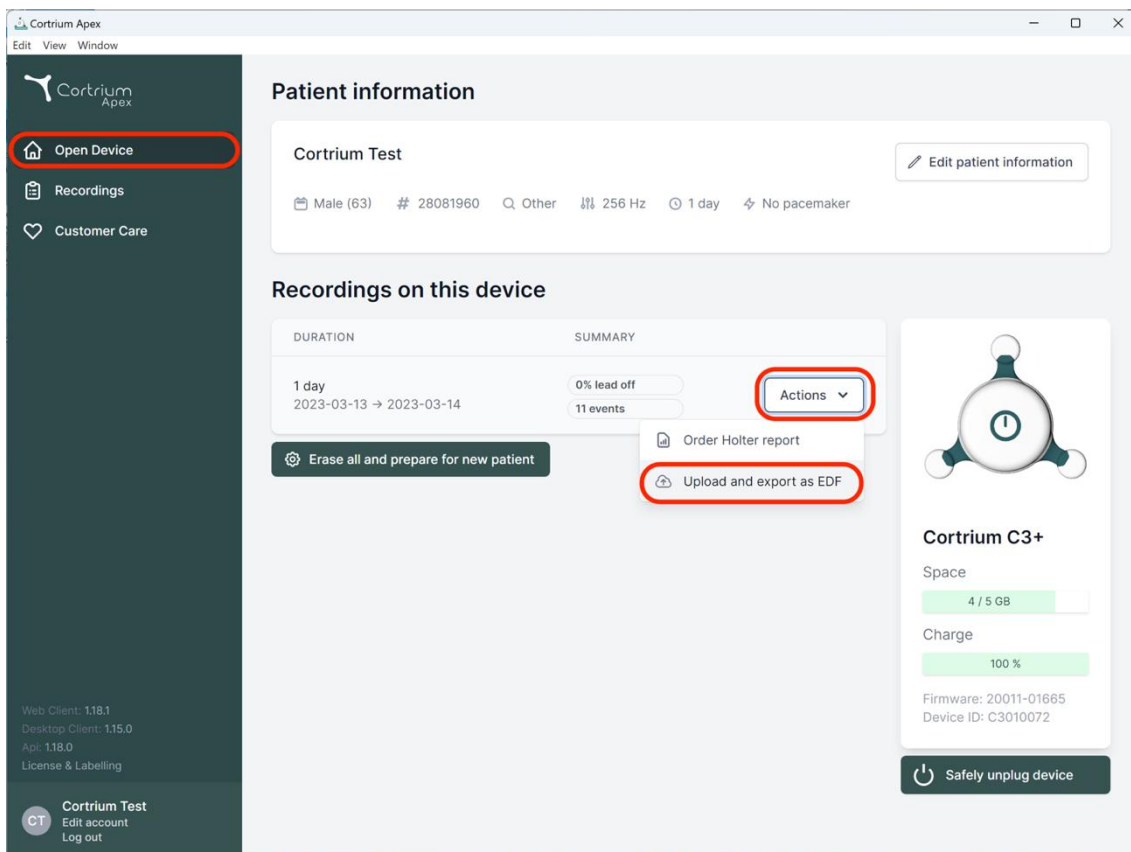
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device' (highlighted with a red circle), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and device settings (256 Hz, 1 day, No pacemaker). Below this is the 'Recordings on this device' section, which displays a recording for '1 day' (2023-03-13 to 2023-03-14) with '0% lead off' and '11 events'. An 'Actions' dropdown menu (highlighted with a red circle) is open, showing options for 'Order Holter report' and 'Upload and export as EDF' (highlighted with a red circle). On the right, there is a device status card for 'Cortrium C3+' showing 'Space' (4/5 GB), 'Charge' (100%), and 'Firmware: 20011-01665'. A 'Safely unplug device' button is at the bottom right. The footer contains version information and user account options for 'Cortrium Test'.

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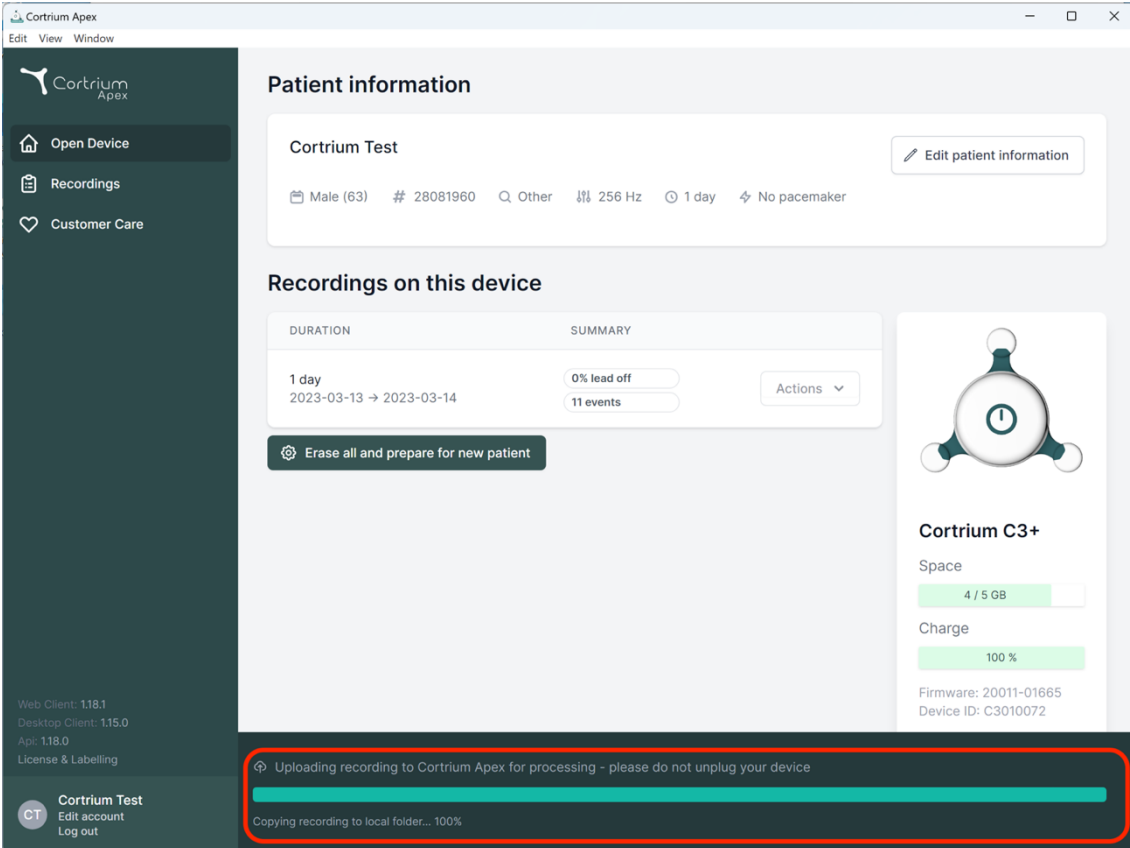Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

### Recordings on this device

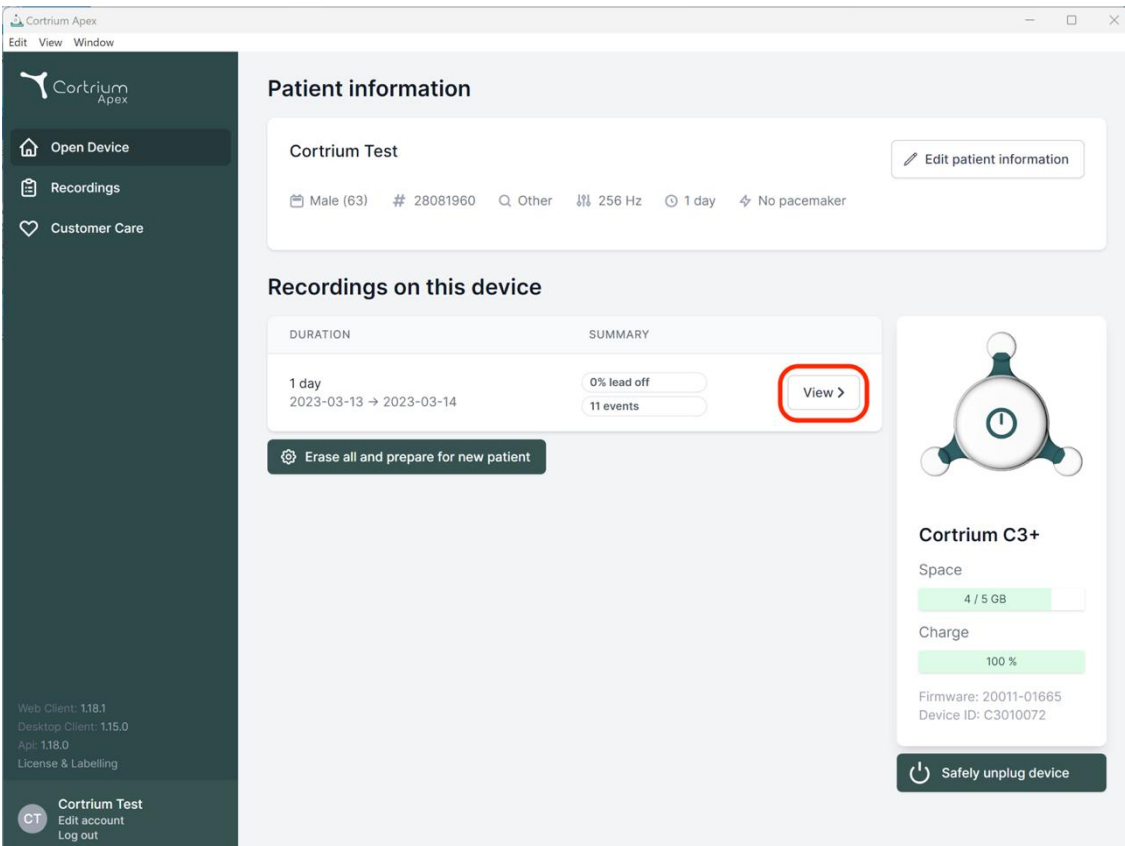
DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">View &gt;</a>

[Erase all and prepare for new patient](#)

**Cortrium C3+**

Space: 4 / 5 GB  
Charge: 100 %  
Firmware: 20011-01665  
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device  
Copying recording to local folder... 100%

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Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

### Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">View &gt;</a>

[Erase all and prepare for new patient](#)

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
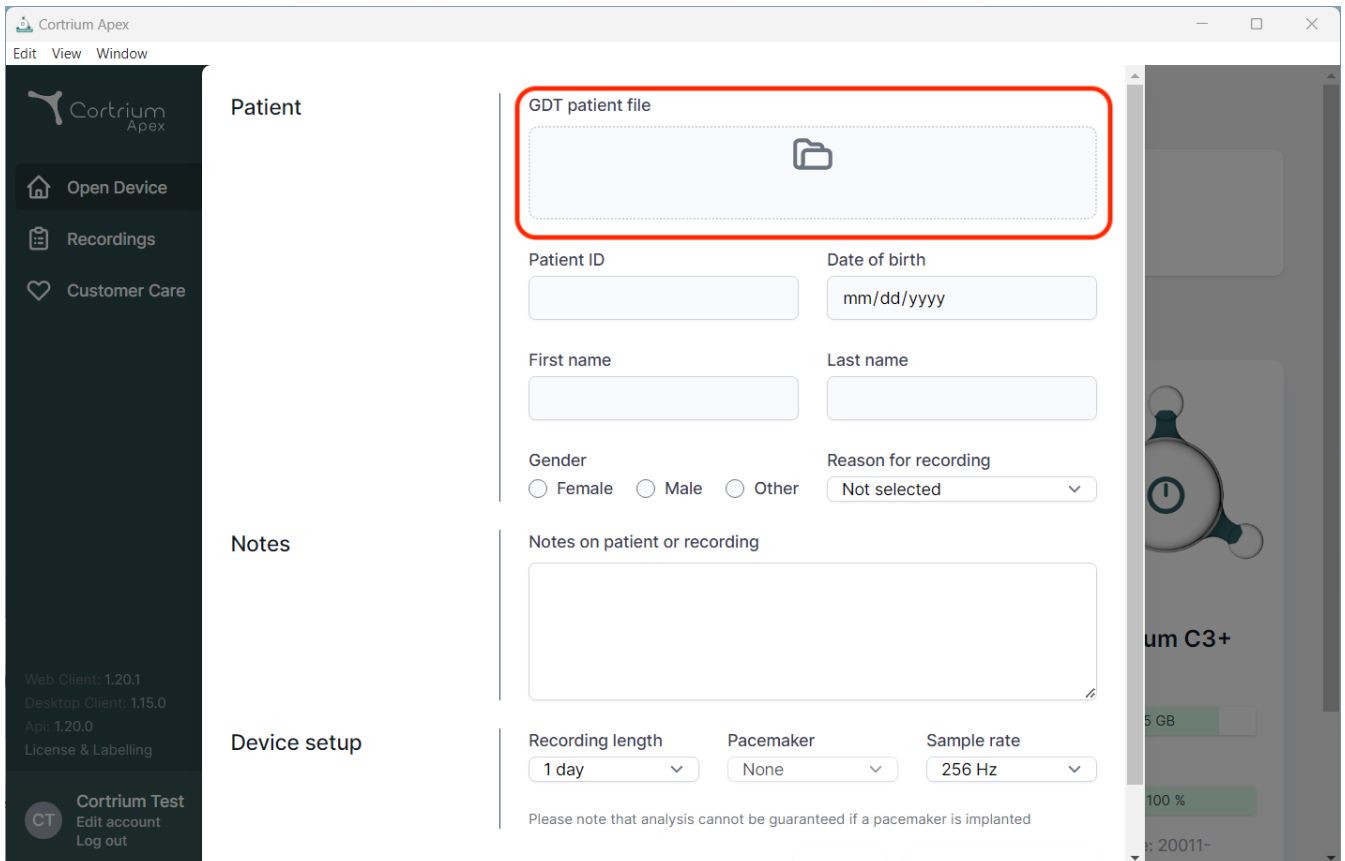
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red box highlights the "GDT patient file" section, which includes a file upload icon. Below this, the form fields are organized as follows:

- Patient ID**: Text input field.
- Date of birth**: Text input field with a placeholder "mm/dd/yyyy".
- First name**: Text input field.
- Last name**: Text input field.
- Gender**: Radio buttons for "Female", "Male", and "Other".
- Reason for recording**: Dropdown menu with "Not selected" as the current option.
- Notes**: A large text area labeled "Notes on patient or recording".
- Device setup**: Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

At the bottom of the form, there is a note: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". The bottom left corner shows the user's name "Cortrium Test" and options to "Edit account" and "Log out".

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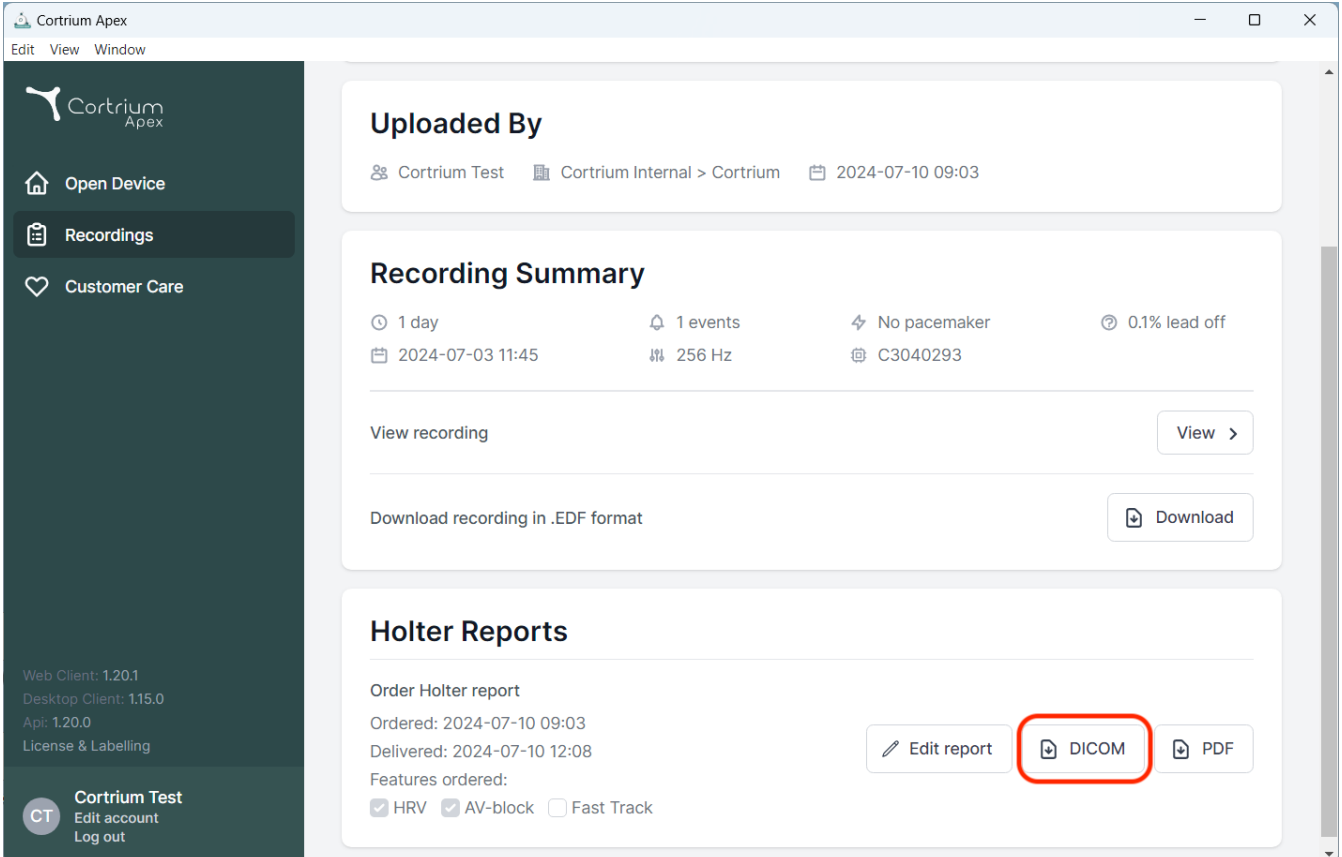The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is divided into three sections:


- Uploaded By:** Shows the user 'Cortrium Test' and the recording date '2024-07-10 09:03'.
- Recording Summary:** Displays key metrics: 1 day duration, 1 event, No pacemaker, and 0.1% lead off. It also shows the recording date '2024-07-03 11:45' and frequency '256 Hz'. There are buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows the report order details: 'Order Holter report', 'Ordered: 2024-07-10 09:03', and 'Delivered: 2024-07-10 12:08'. It lists ordered features: HRV (checked), AV-block (checked), and Fast Track (unchecked). At the bottom right, there are buttons for 'Edit report', 'DICOM' (highlighted with a red box), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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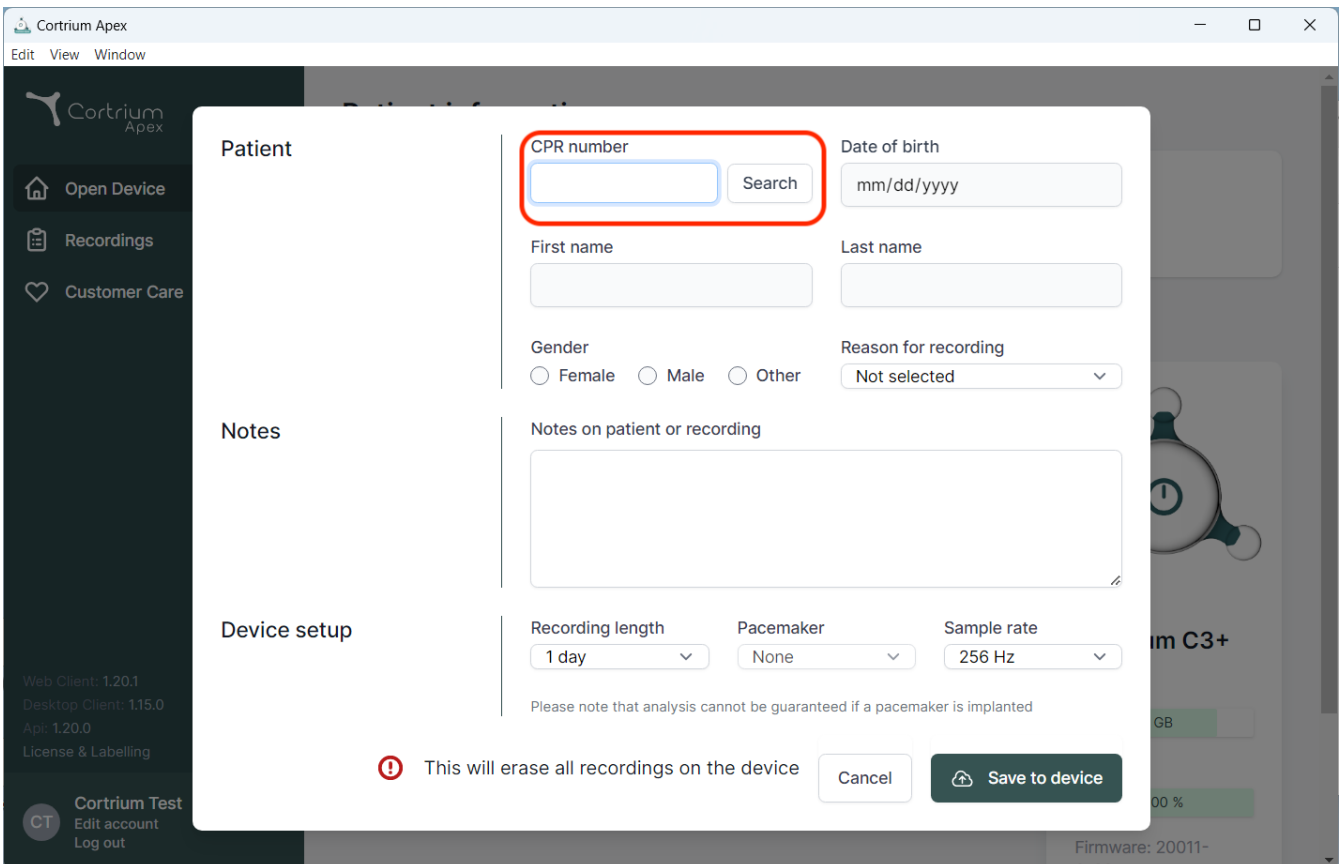Recordings

Customer Care

Web Client: 1.20.1  
Desktop Client: 1.15.0  
App: 1.20.0  
License & Labelling

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**!** This will erase all recordings on the device

Cancel

09/2024

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Taani  
E-post: [info@cortrium.com](mailto:info@cortrium.com) • Veebisait: [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033