Cortrium Apex 1.20

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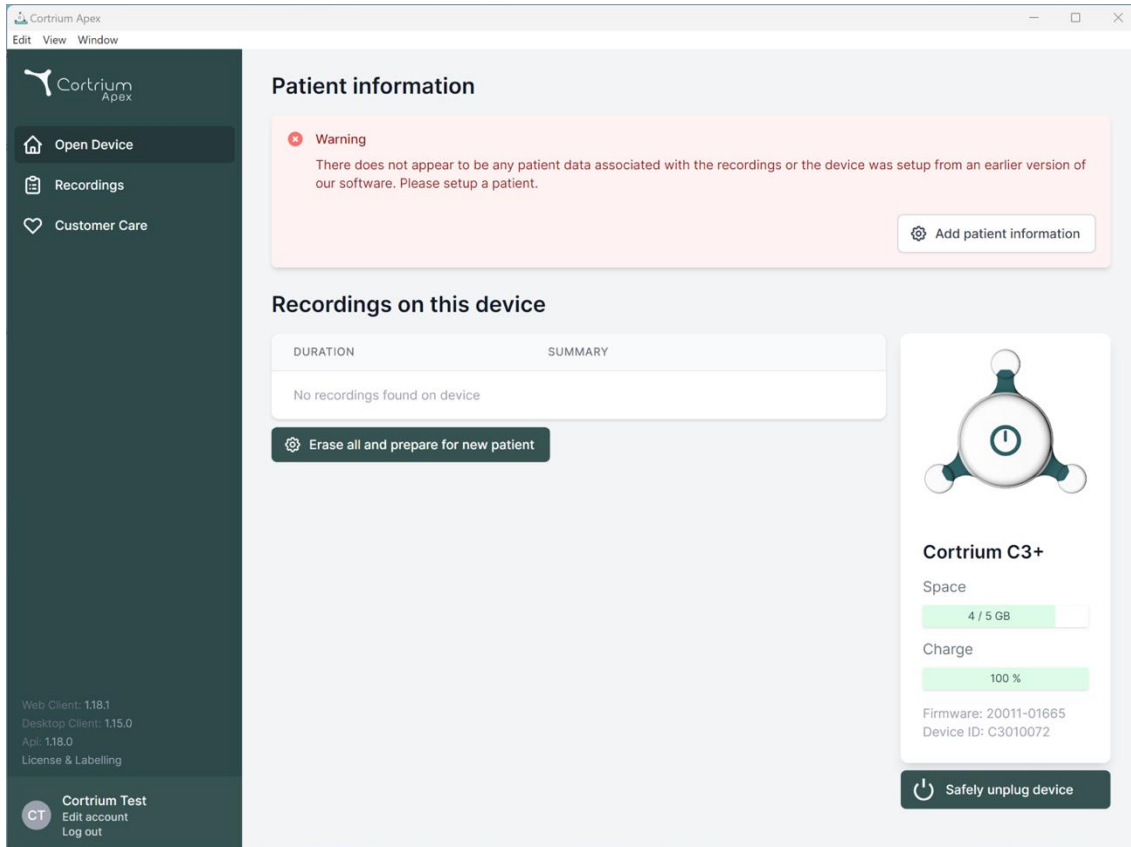
Cortrium Test
Edit account
Log out

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.

Add patient information

Recordings on this device

| DURATION | SUMMARY |
|-------------------------------|---------|
| No recordings found on device | |

Erase all and prepare for new patient

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Cortrium Apex
Edit View Window

Cortrium Apex

Open Device
Recordings
Customer Care

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: Female Male Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

⚠ This will erase all recordings on the device

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
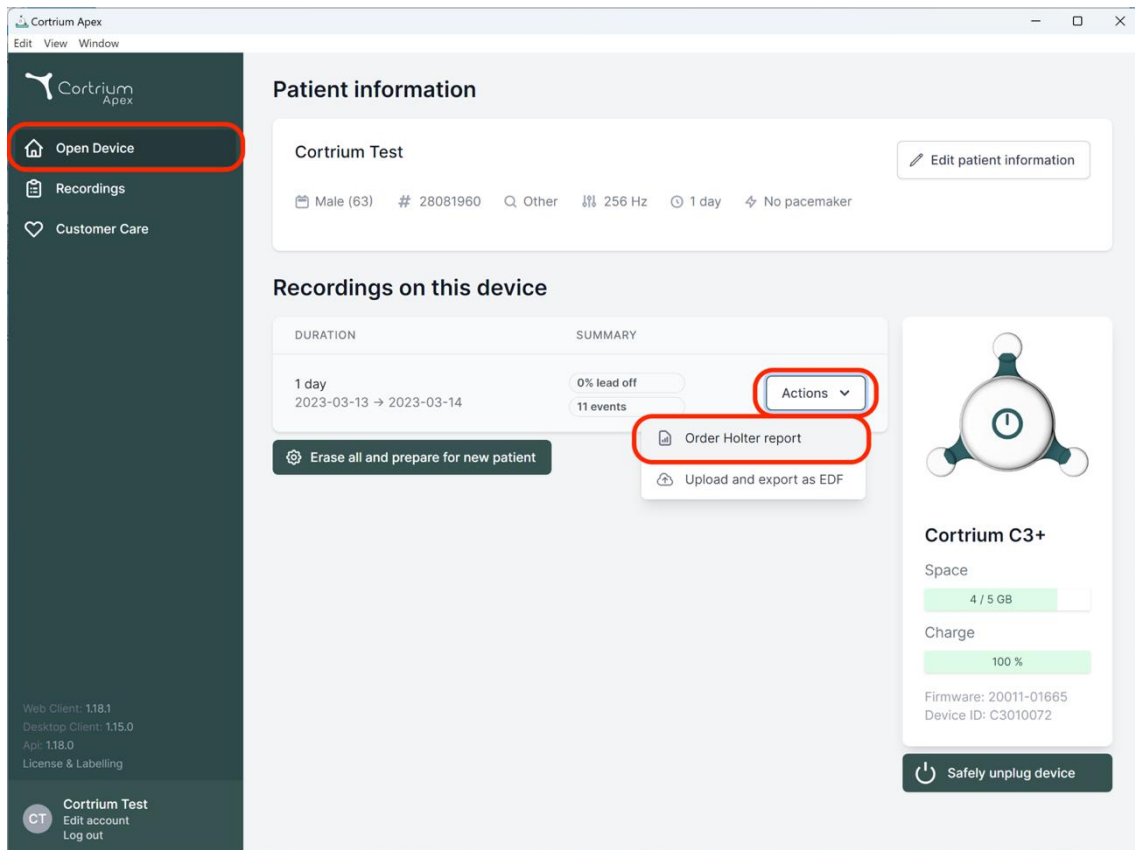
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device' (highlighted with a red circle), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and test duration (1 day, 2023-03-13 to 2023-03-14). Below this is a table of recordings with columns for 'DURATION' and 'SUMMARY'. The 'SUMMARY' column shows '0% lead off' and '11 events'. An 'Actions' dropdown menu is open, with 'Order Holter report' (highlighted with a red circle) and 'Upload and export as EDF' visible. On the right, there is a device status card for 'Cortrium C3+' showing 'Space: 4 / 5 GB' and 'Charge: 100%'. At the bottom right of the device card is a 'Safely unplug device' button. The bottom of the sidebar shows the user's name 'Cortrium Test' and options to 'Edit account' and 'Log out'.

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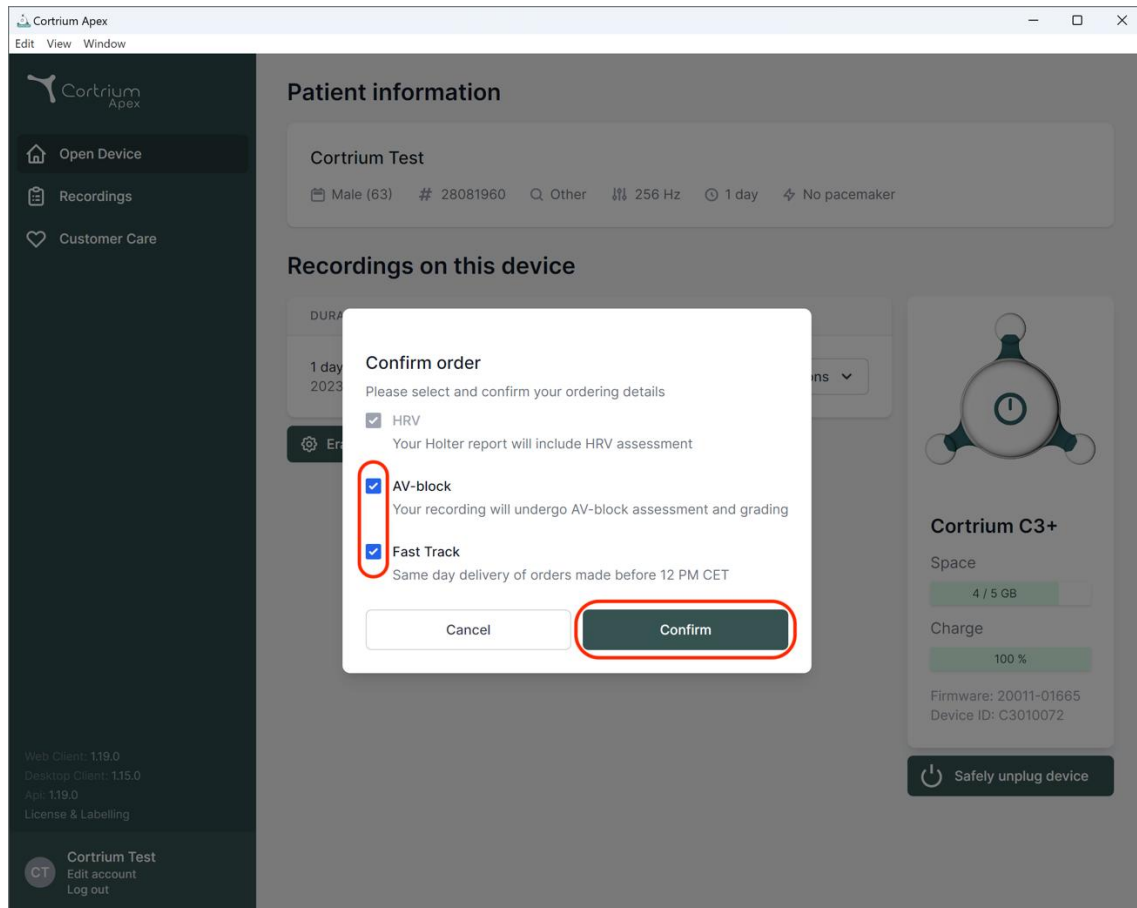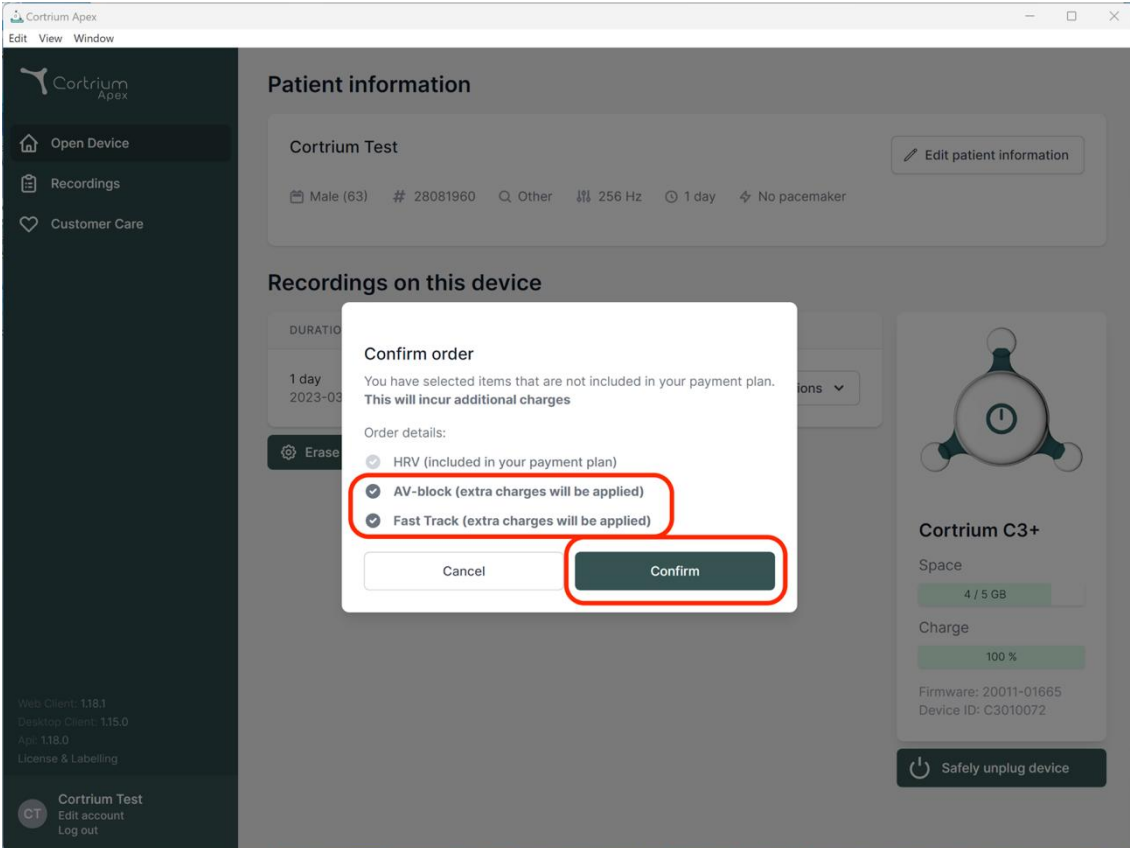
Erase

Confirm order

You have selected items that are not included in your payment plan. This will incur additional charges

Order details:

- HRV (included in your payment plan)
- AV-block (extra charges will be applied)
- Fast Track (extra charges will be applied)

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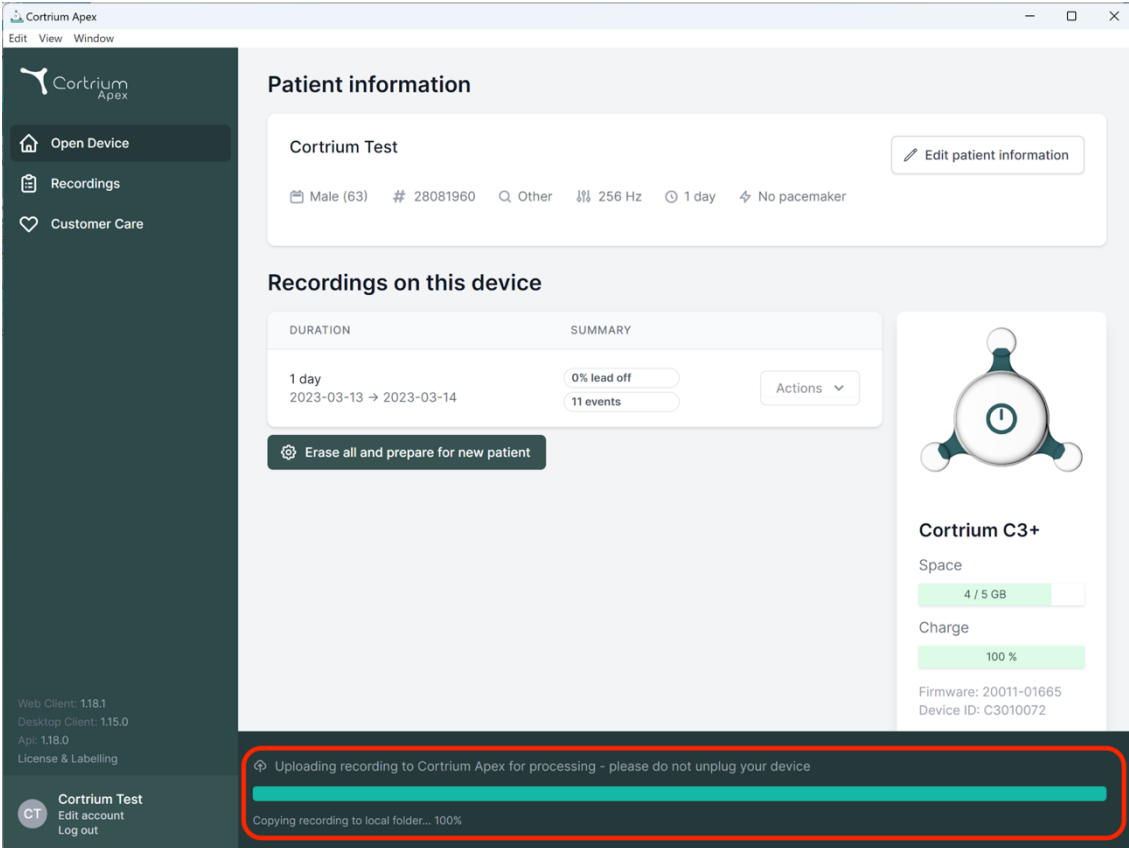
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The main content area is divided into sections: Patient information for 'Cortrium Test' (Male, 63, ID 28081960, 1 day recording, No pacemaker), Recordings on this device (1 day, 2023-03-13 to 2023-03-14, 0% lead off, 11 events), and device status for Cortrium C3+ (Space: 4/5 GB, Charge: 100%, Firmware: 20011-01665, Device ID: C3010072). A red box highlights a progress bar at the bottom of the interface, indicating the upload of a recording to Cortrium Apex for processing (100%) and copying to a local folder (100%).

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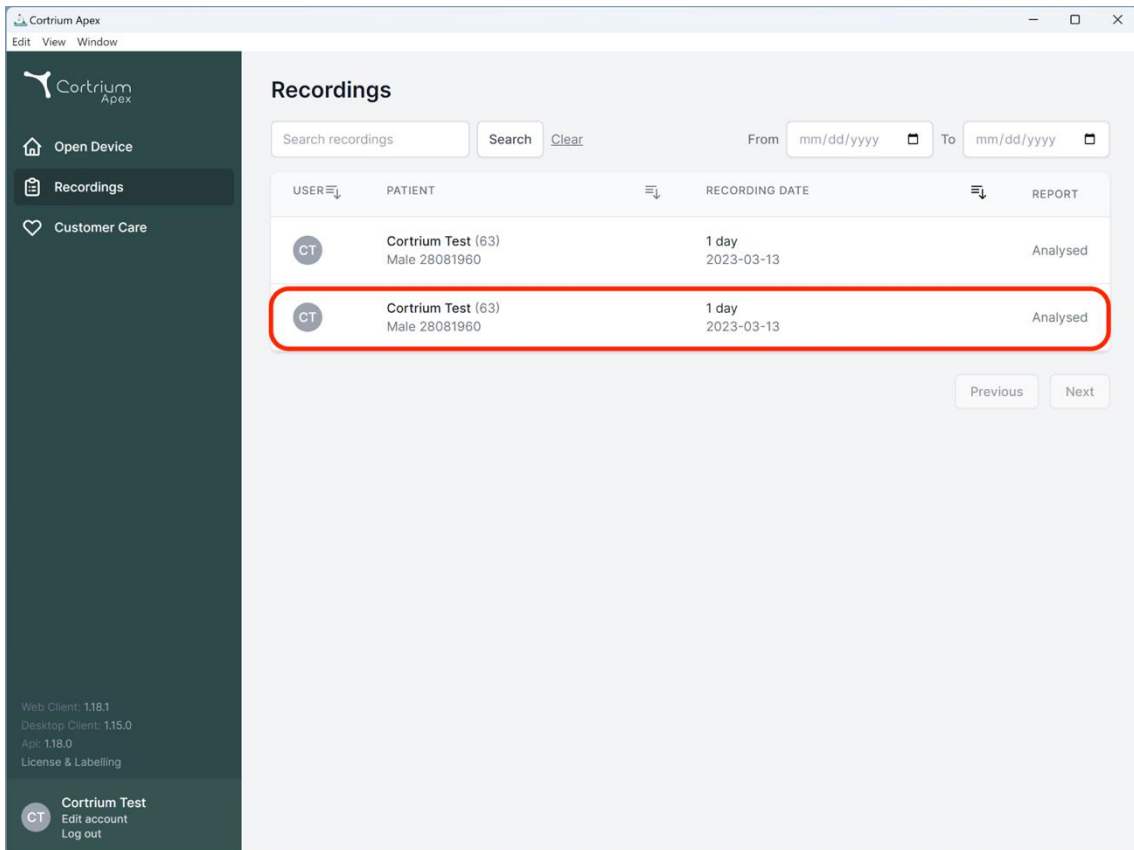
The screenshot shows the Cortrium Apex web application interface. The main content area is titled "Recordings" and features a search bar with "Search" and "Clear" buttons, and date selection fields for "From" and "To". Below the search bar is a table with the following columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains two rows of data, both of which are highlighted with a red border:

| USER | PATIENT | RECORDING DATE | REPORT |
|------|-------------------------------------|---------------------|----------|
| CT | Cortrium Test (63) Male 28081960 | 1 day 2023-03-13 | Analysed |
| CT | Cortrium Test (63) Male 28081960 | 1 day 2023-03-13 | Analysed |

At the bottom right of the table, there are "Previous" and "Next" navigation buttons. The sidebar on the left contains the Cortrium Apex logo and navigation options: "Open Device", "Recordings", and "Customer Care". The footer of the sidebar displays version information: "Web Client: 1.18.1", "Desktop Client: 1.15.0", "Api: 1.18.0", and "License & Labelling". At the bottom of the sidebar, there is a user profile section for "Cortrium Test" with options to "Edit account" and "Log out".

Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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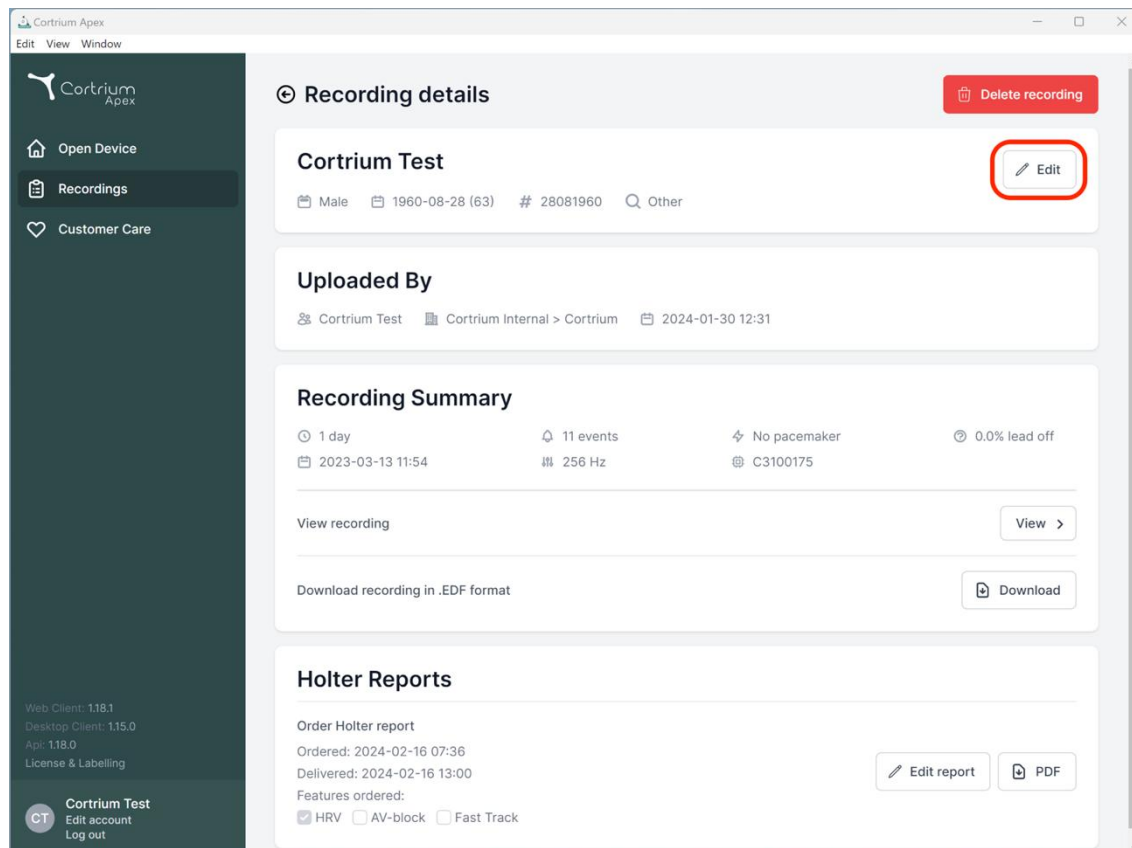
Recording details Delete recording

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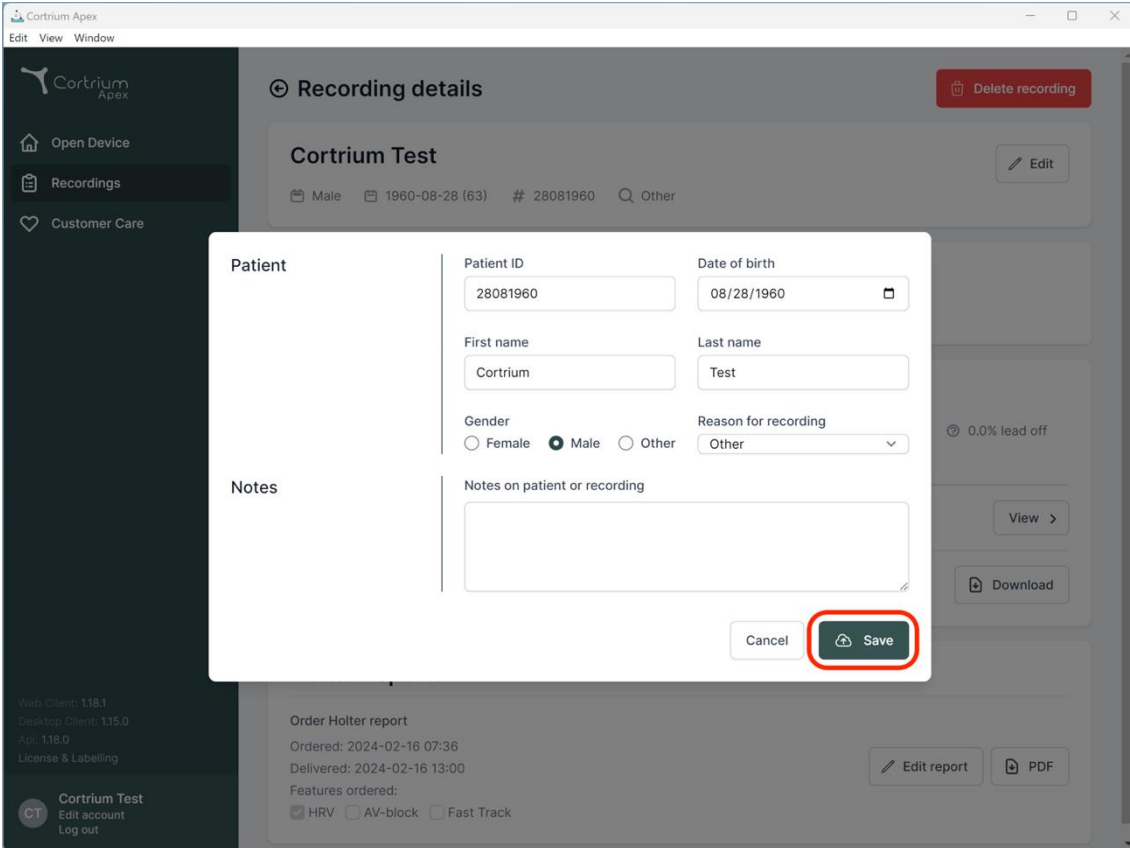
Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track Edit report PDF

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Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: Female Male Other Reason for recording: Other

Notes

Notes on patient or recording

Cancel Save

Web Client: 118.1 Desktop Client: 115.0 App: 118.0 License & Labelling

CT Cortrium Test Edit account Log out

Order Holter report

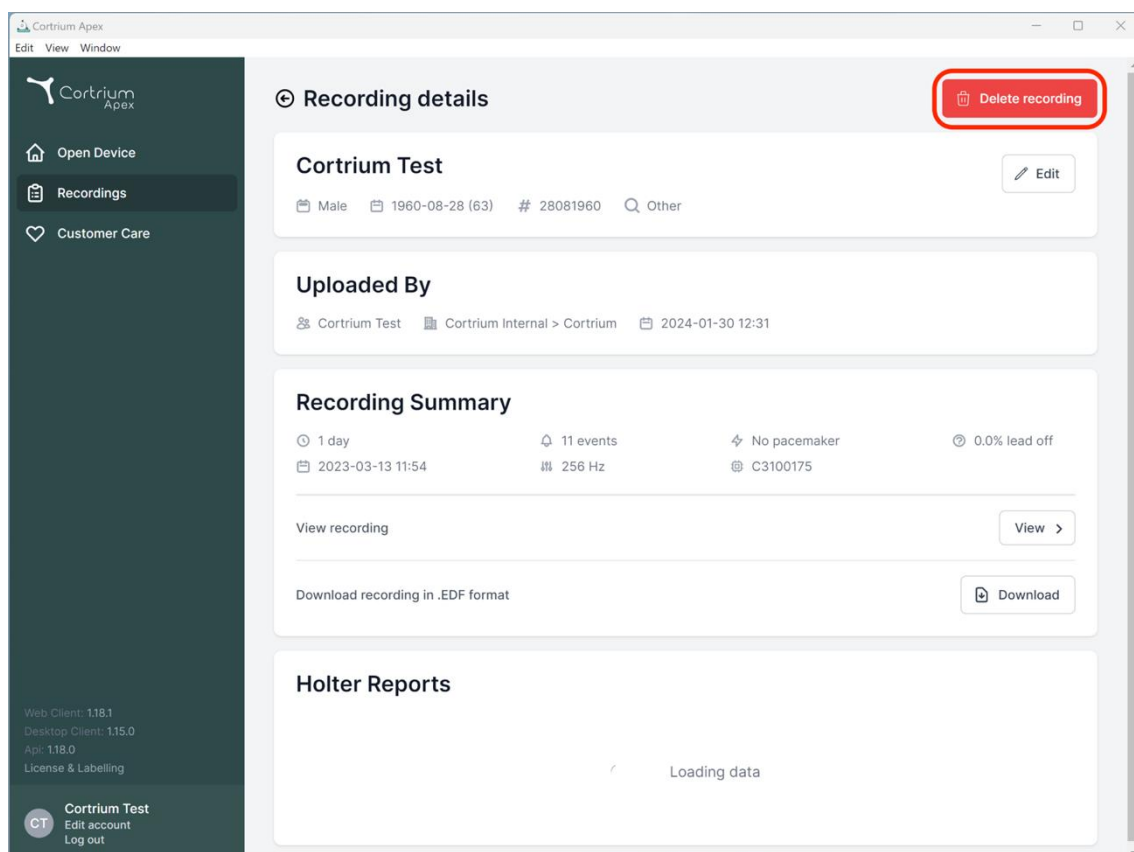
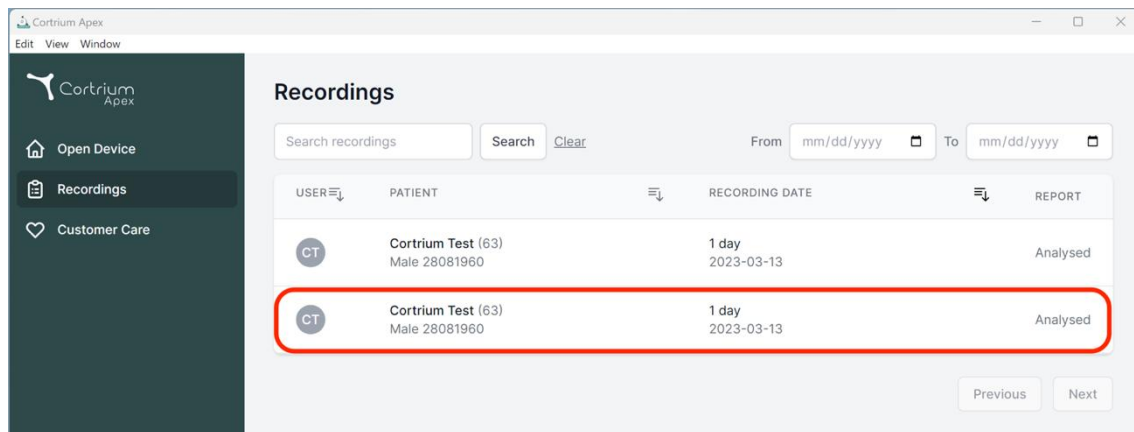
Ordered: 2024-02-16 07:36 Delivered: 2024-02-16 13:00

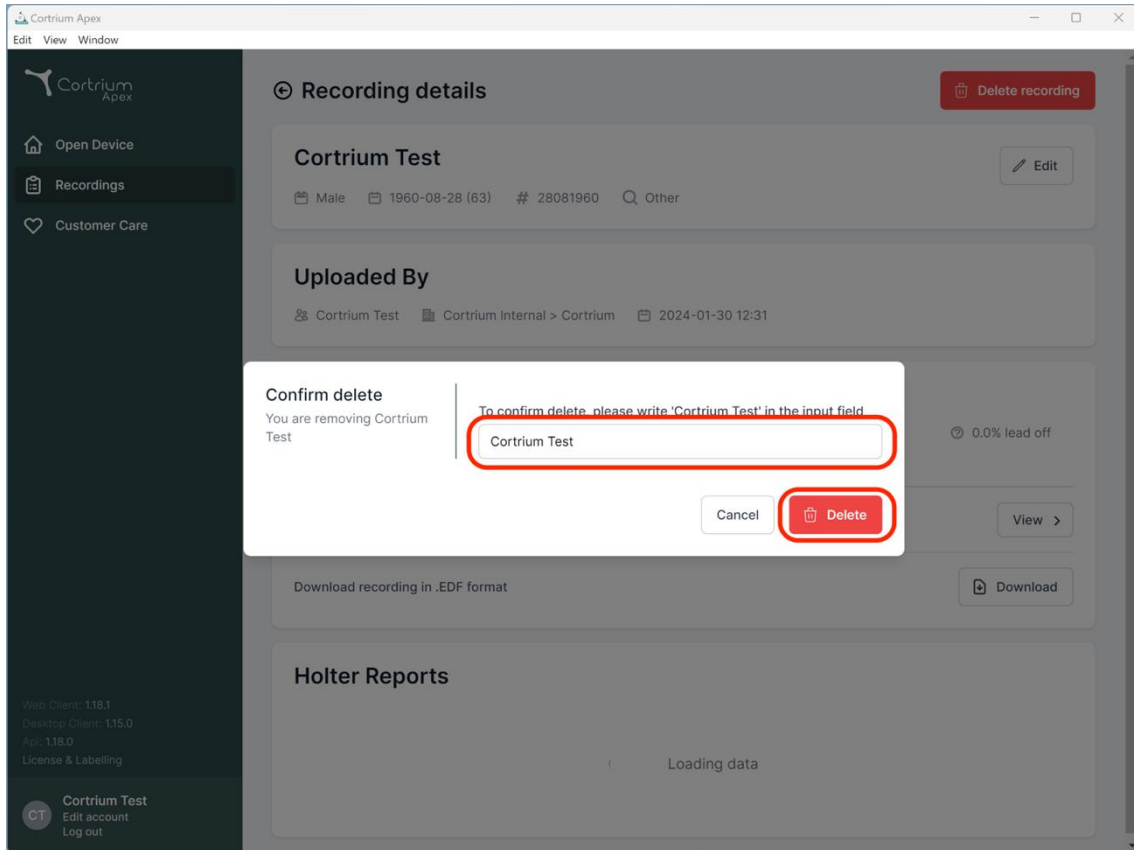
Features ordered: HRV AV-block Fast Track

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The screenshot shows the Cortrium Apex web application interface. A 'Confirm delete' dialog box is open in the foreground, overlaid on the 'Recording details' page. The dialog box contains the following text and elements:

- Confirm delete**
- You are removing Cortrium Test
- To confirm delete, please write 'Cortrium Test' in the input field.
- An input field containing the text 'Cortrium Test'.
- Buttons for 'Cancel' and 'Delete'.

The background page, 'Recording details', shows the following information:

- Recording details** (with a 'Delete recording' button)
- Cortrium Test** (with an 'Edit' button)
- Male, 1960-08-28 (63), # 28081960, Other
- Uploaded By**
- Cortrium Test, Cortrium Internal > Cortrium, 2024-01-30 12:31
- 0.0% lead off
- Download recording in .EDF format (with a 'Download' button)
- Holter Reports**
- Loading data

The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. At the bottom of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The top of the browser window shows 'Cortrium Apex' and standard window controls.

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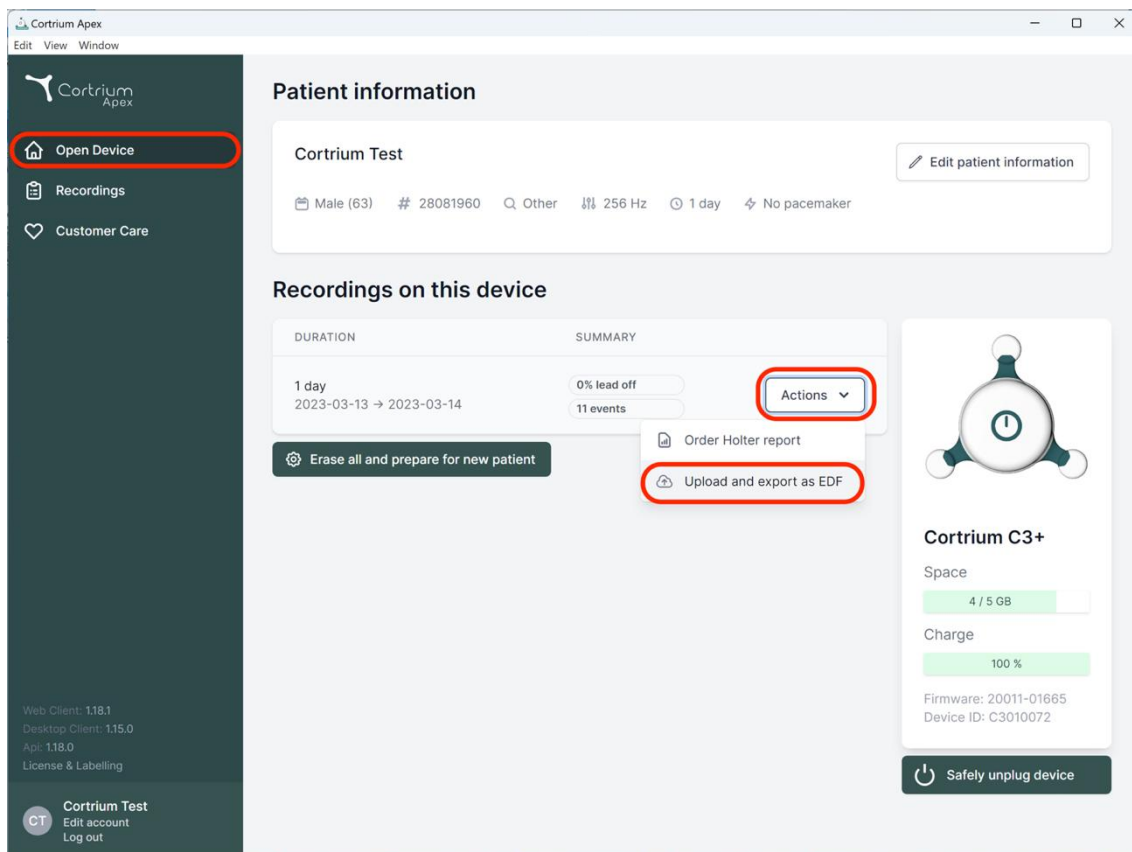
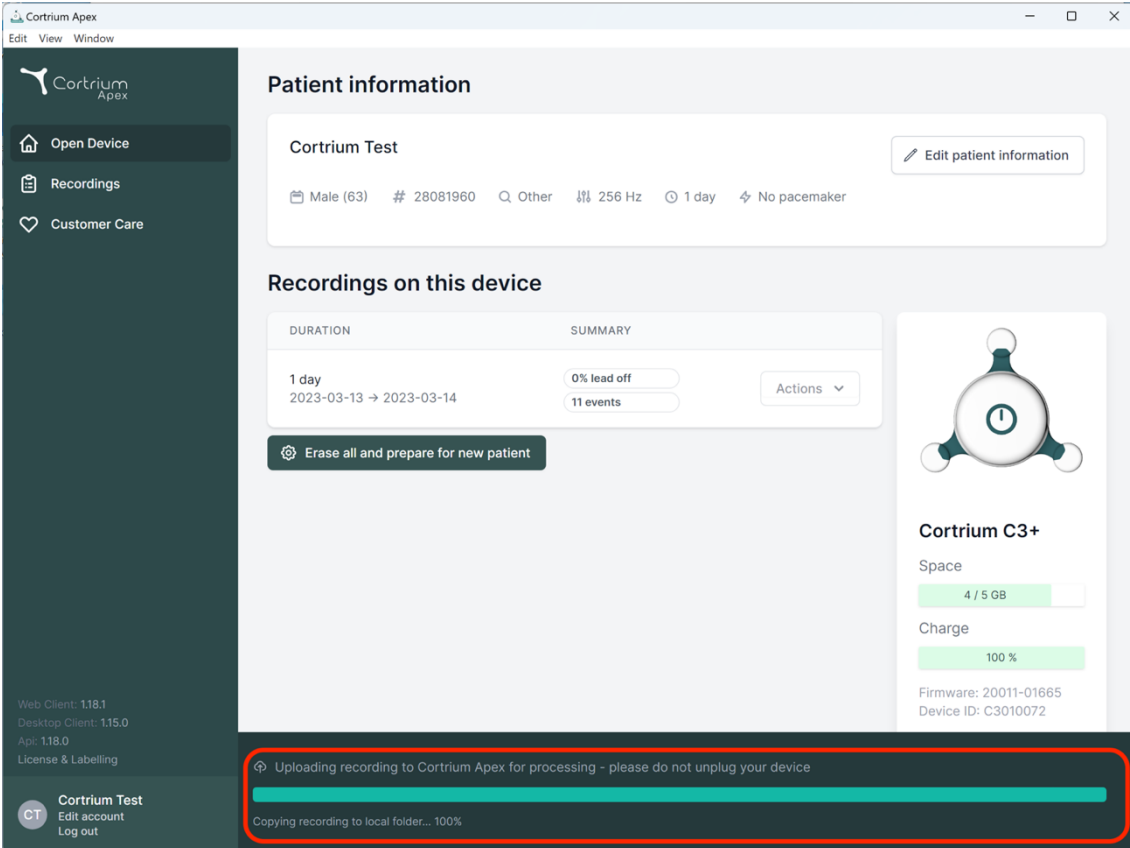
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

| DURATION | SUMMARY |
|----------------------------------|--------------------------|
| 1 day 2023-03-13 → 2023-03-14 | 0% lead off 11 events |

[Erase all and prepare for new patient](#)

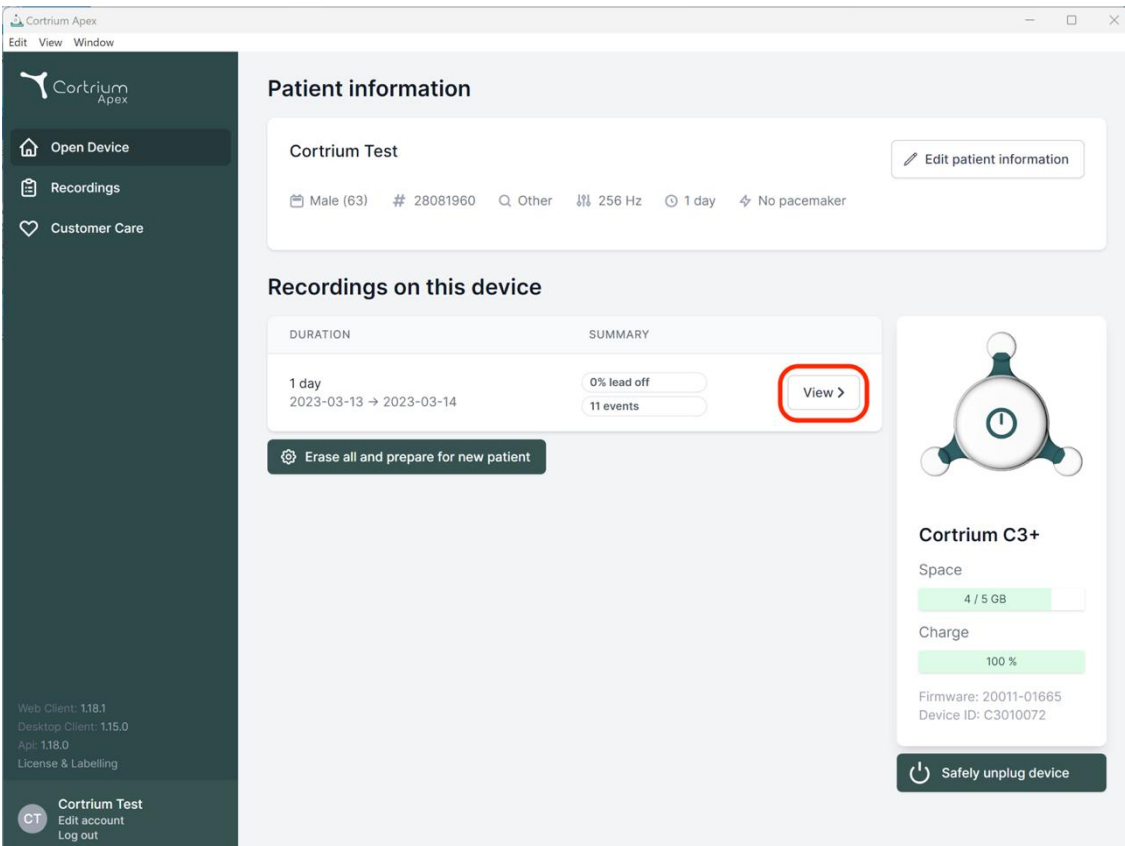
Cortrium C3+

Space: 4 / 5 GB
Charge: 100 %
Firmware: 20011-01665
Device ID: C3010072

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

Cortrium Test
CT Edit account Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device
Copying recording to local folder... 100%

Cortrium Apex
Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

| DURATION | SUMMARY |
|----------------------------------|--------------------------|
| 1 day 2023-03-13 → 2023-03-14 | 0% lead off 11 events |

[Erase all and prepare for new patient](#)

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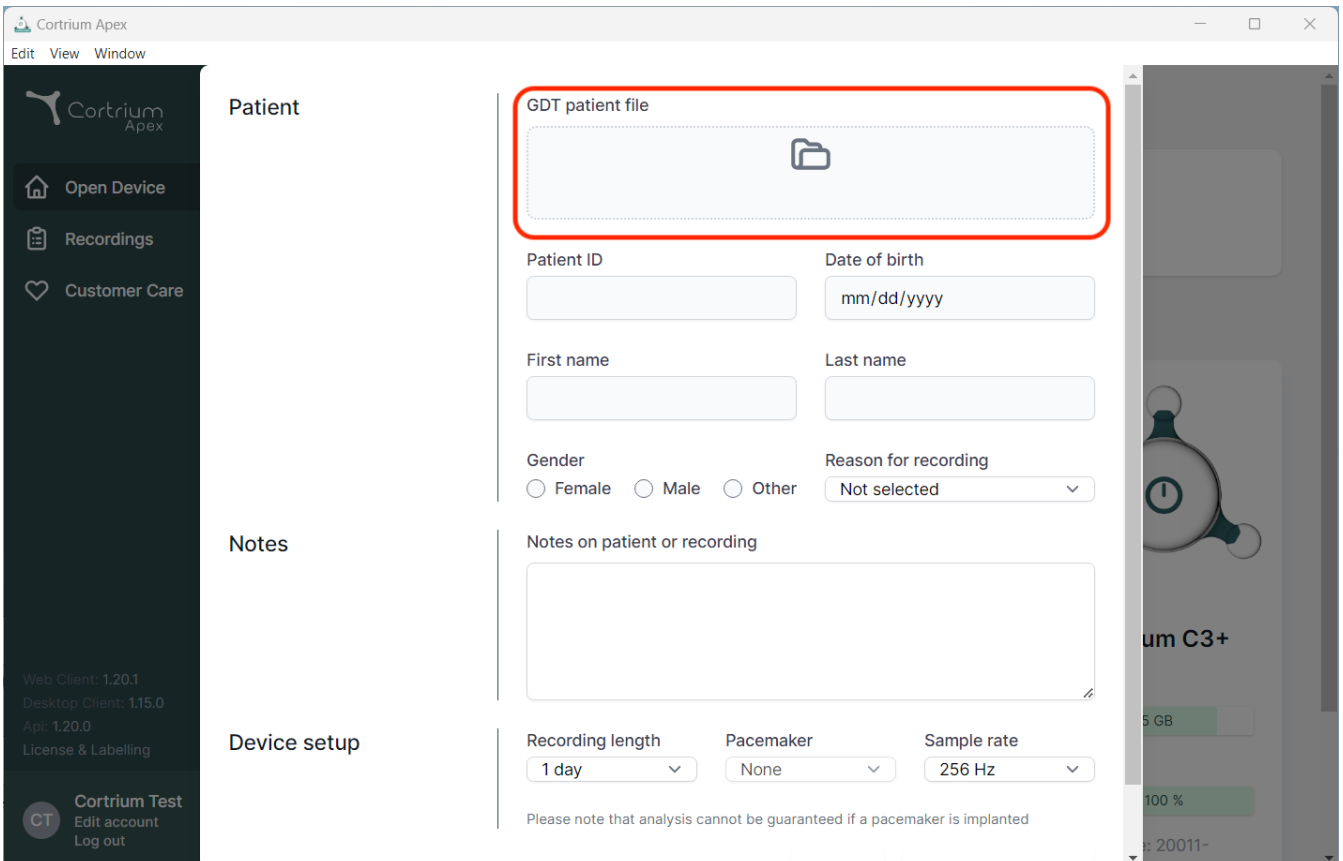
The screenshot shows the Cortrium Apex web application interface. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a form for entering patient information. The 'GDT patient file' field is highlighted with a red box. Below it are fields for Patient ID, Date of birth (mm/dd/yyyy), First name, Last name, Gender (Female, Male, Other), and Reason for recording (Not selected). The Notes section has a text area for 'Notes on patient or recording'. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. The left sidebar contains navigation options: Open Device, Recordings, Customer Care, and user information for 'Cortrium Test'.

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Web Client: 1.20.1
Desktop Client: 1.15.0
App: 1.20.0
License & Labelling

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
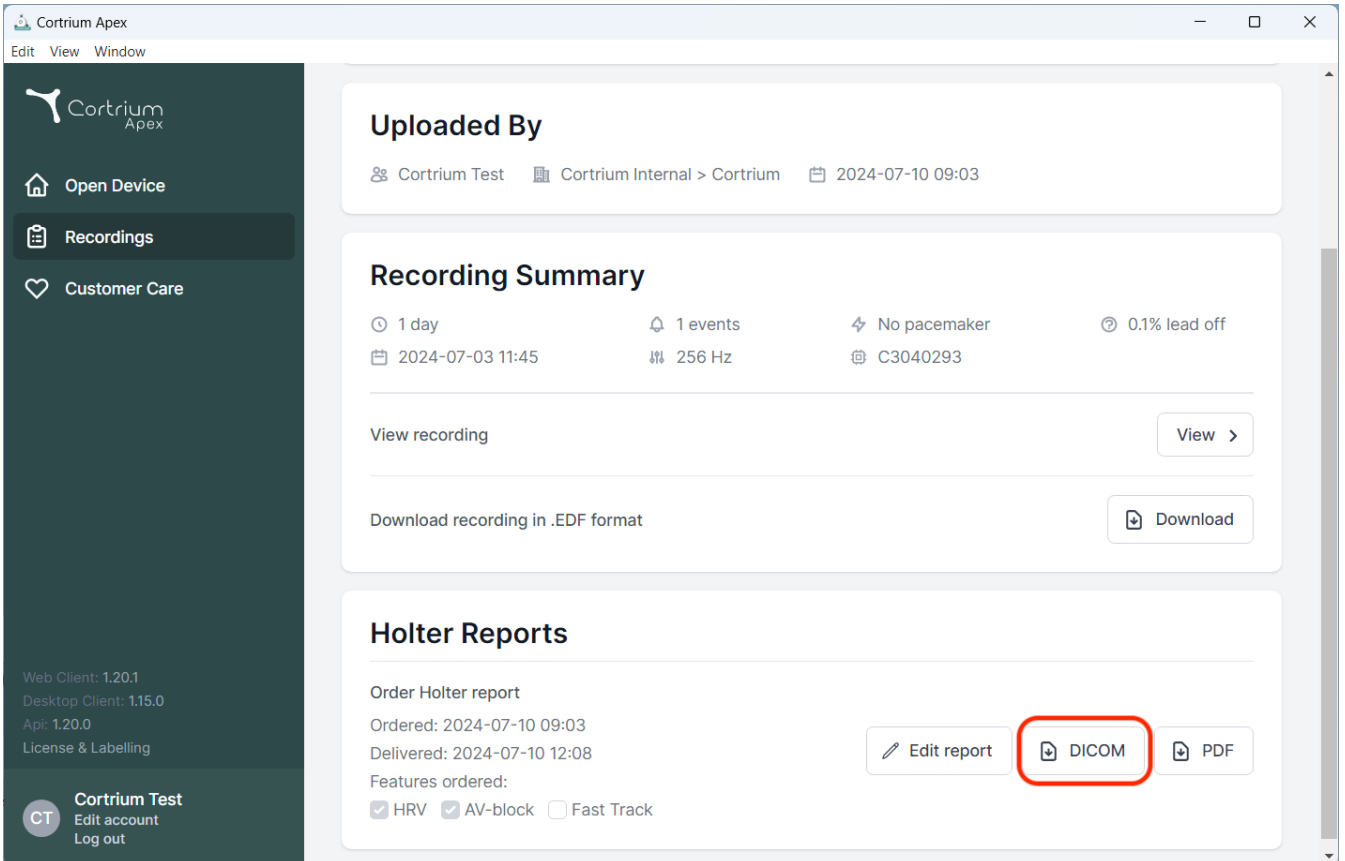
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is divided into three sections:

- Uploaded By:** Shows the user 'Cortrium Test' and the date '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: 1 day duration, 1 event, No pacemaker, and 0.1% lead off. It includes a 'View recording' button and a 'Download recording in .EDF format' button.
- Holter Reports:** Shows the report order details: Ordered on 2024-07-10 09:03 and Delivered on 2024-07-10 12:08. It lists ordered features: HRV (checked), AV-block (checked), and Fast Track (unchecked). Three buttons are visible: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. Version information for Web Client (1.20.1), Desktop Client (1.15.0), and Api (1.20.0) is also present.

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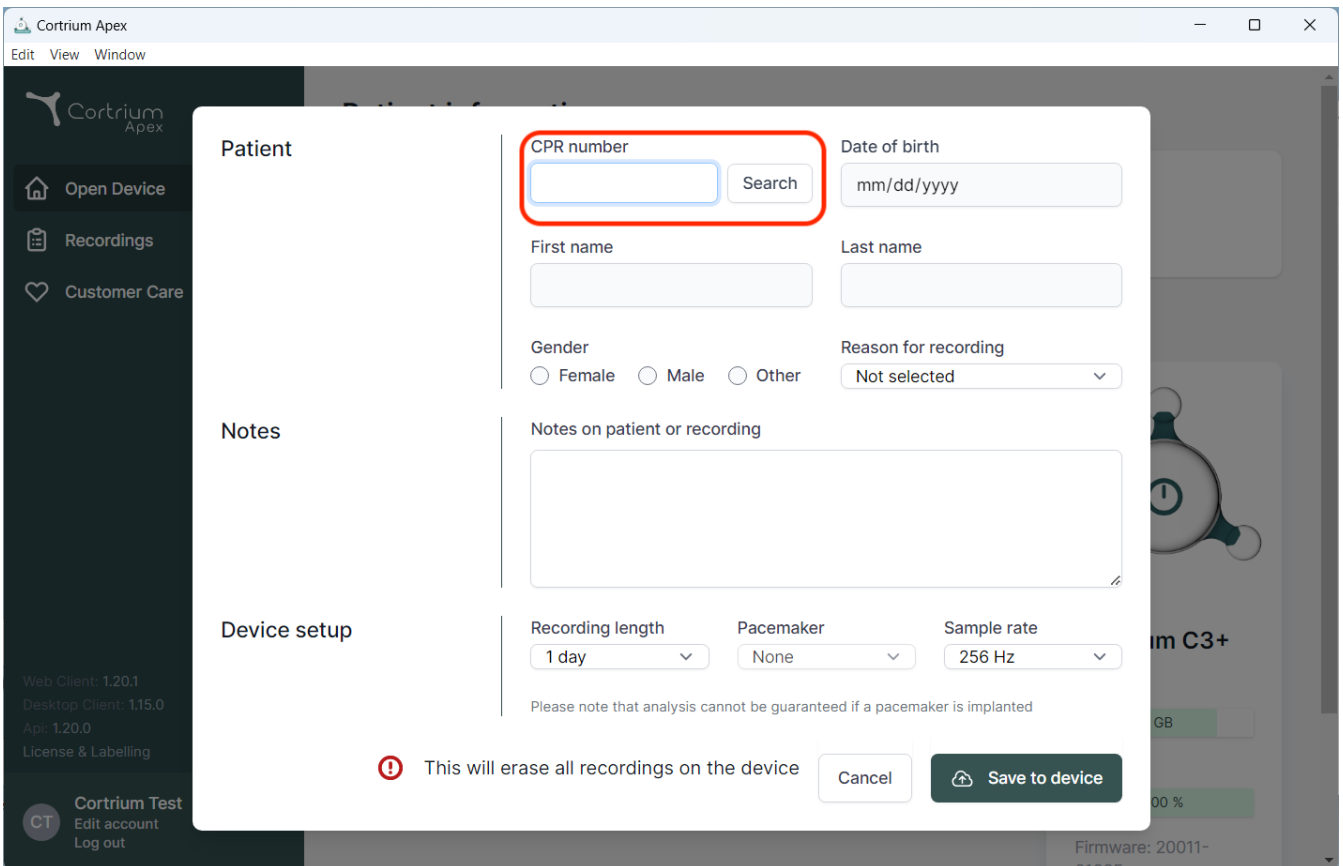Recordings

Customer Care

Web Client: 1.20.1
Desktop Client: 1.15.0
App: 1.20.0
License & Labelling

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! This will erase all recordings on the device

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