# Cortrium Apex 1.20

## Mode d'emploi

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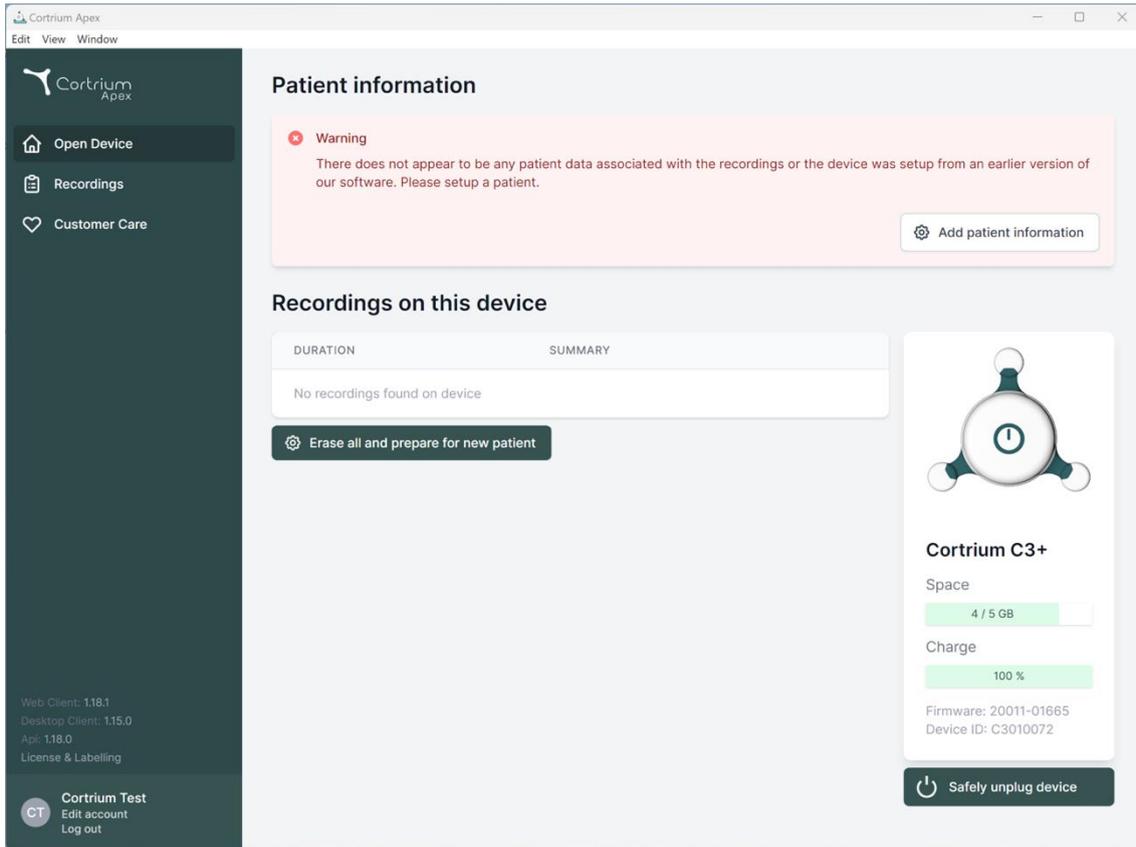
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, it displays version information (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, License & Labelling) and user options for 'Cortrium Test' (Edit account, Log out). The main content area is titled 'Patient information' and features a warning message: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' Below this is a table for 'Recordings on this device' with columns for 'DURATION' and 'SUMMARY', showing 'No recordings found on device'. A button 'Erase all and prepare for new patient' is located below the table. On the right, a device status card for 'Cortrium C3+' shows 'Space' at 4 / 5 GB and 'Charge' at 100%. It also lists 'Firmware: 20011-01665' and 'Device ID: C3010072', with a 'Safely unplug device' button at the bottom.

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Cortrium Apex  
Edit View Window

Cortrium Apex

Open Device  
Recordings  
Customer Care

### Patient information

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

**Patient**

Patient ID: 28081960  
Date of birth: 08/28/1960

First name: Cortrium  
Last name: Test

Gender:  Female  Male  Other  
Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day  
Pacemaker: None  
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

**⚠ This will erase all recordings on the device**

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
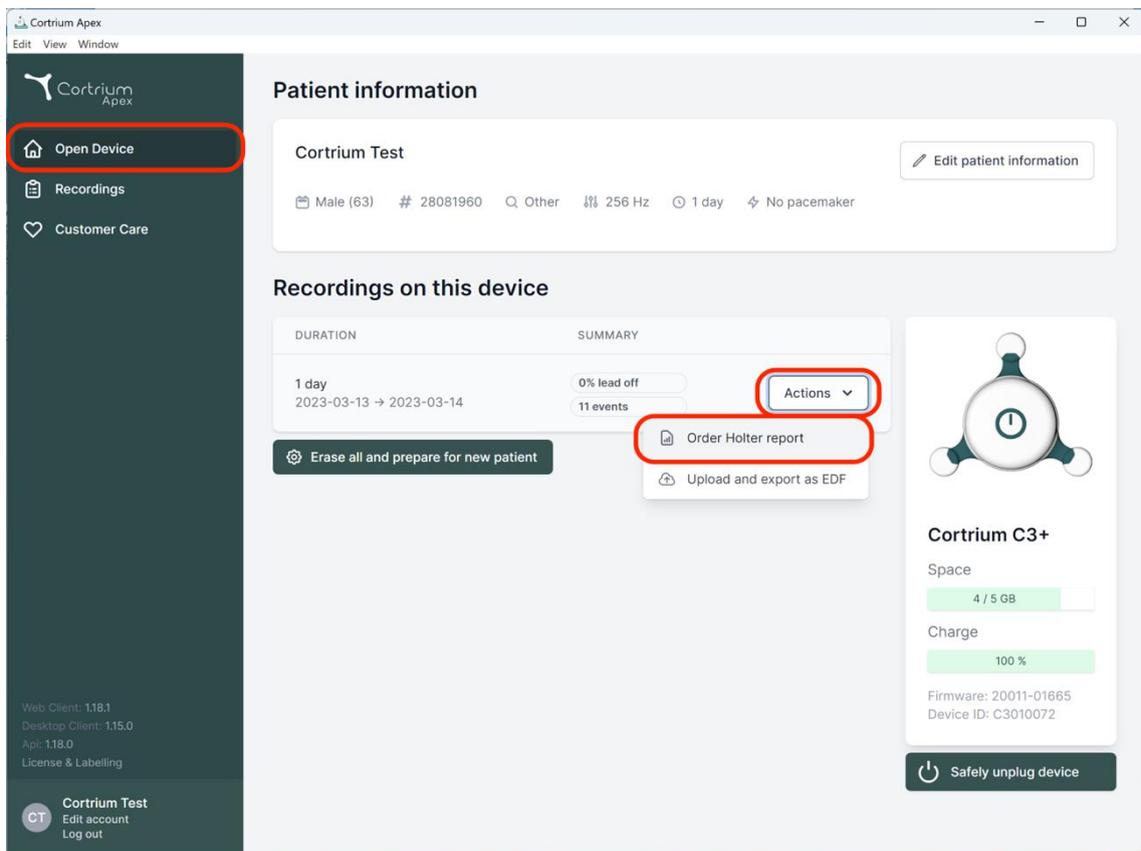
The screenshot displays the Cortrium Apex web application interface. On the left, a dark sidebar contains navigation options: 'Open Device' (highlighted with a red box), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including patient demographics and device settings. Below this, the 'Recordings on this device' section features a table with columns for 'DURATION' and 'SUMMARY'. A recording entry for '1 day' (2023-03-13 to 2023-03-14) is shown with a '0% lead off' and '11 events'. An 'Actions' dropdown menu is open, with 'Order Holter report' (highlighted with a red box) and 'Upload and export as EDF' visible. On the right, a device status card for 'Cortrium C3+' shows 'Space' at 4/5 GB and 'Charge' at 100%. A 'Safely unplug device' button is at the bottom right. The footer includes version information and user account options for 'Cortrium Test'.

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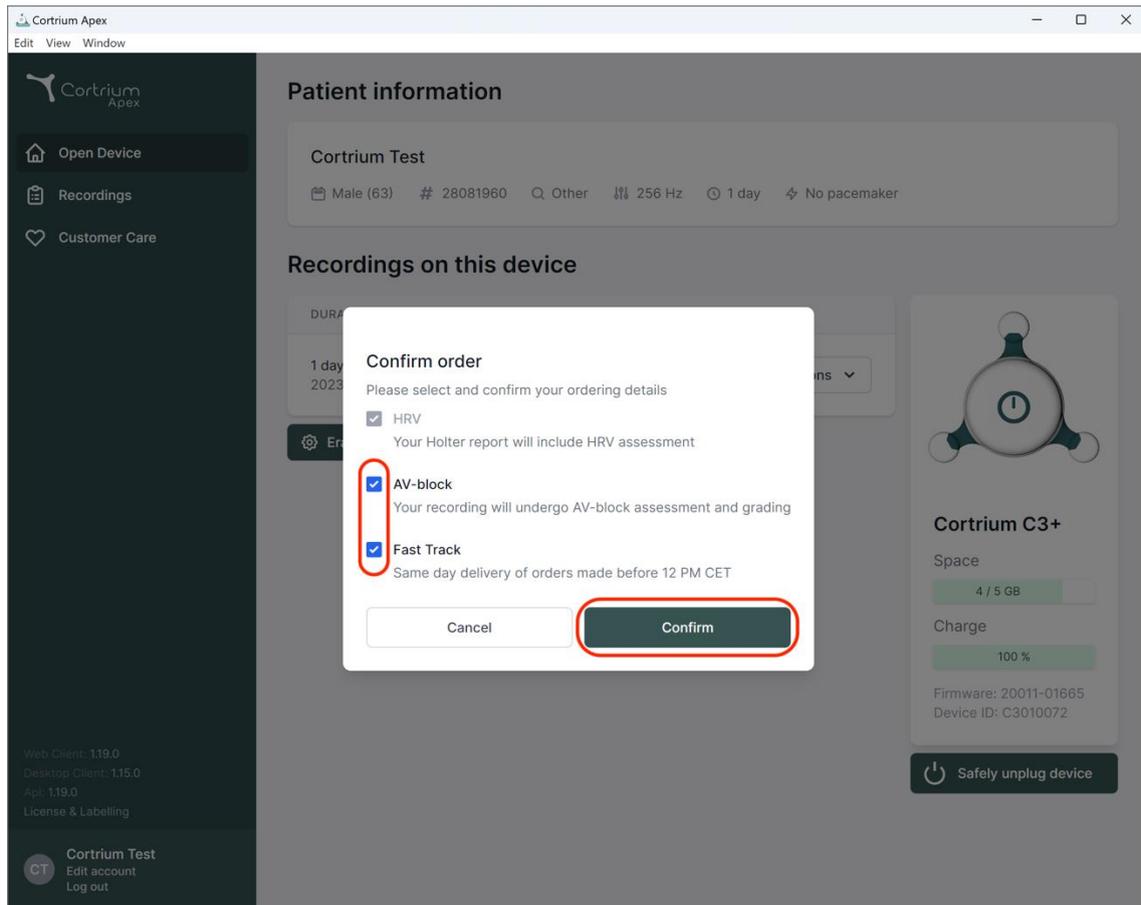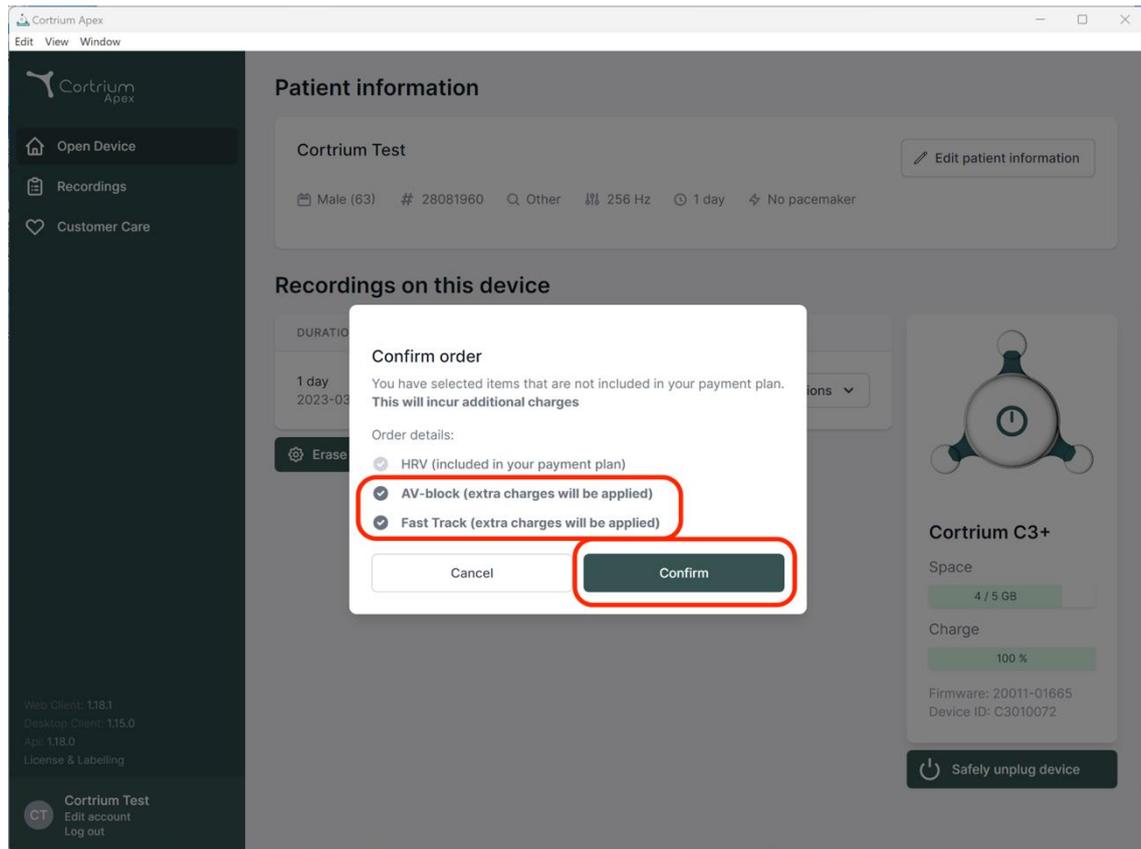
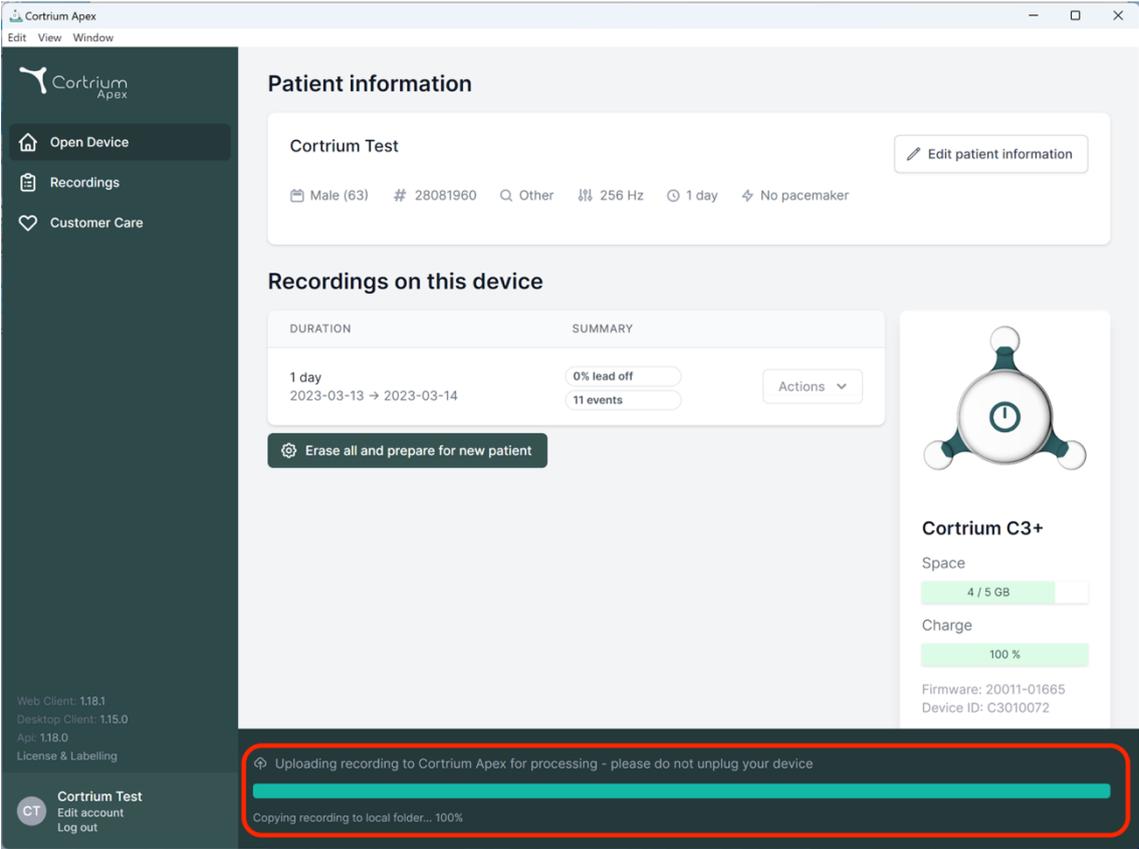
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, a 'Recordings on this device' section shows a summary of 0% lead off and 11 events. To the right, a 'Cortrium C3+' device status card shows 4/5 GB space and 100% charge. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. The sidebar also contains version information and user account options.

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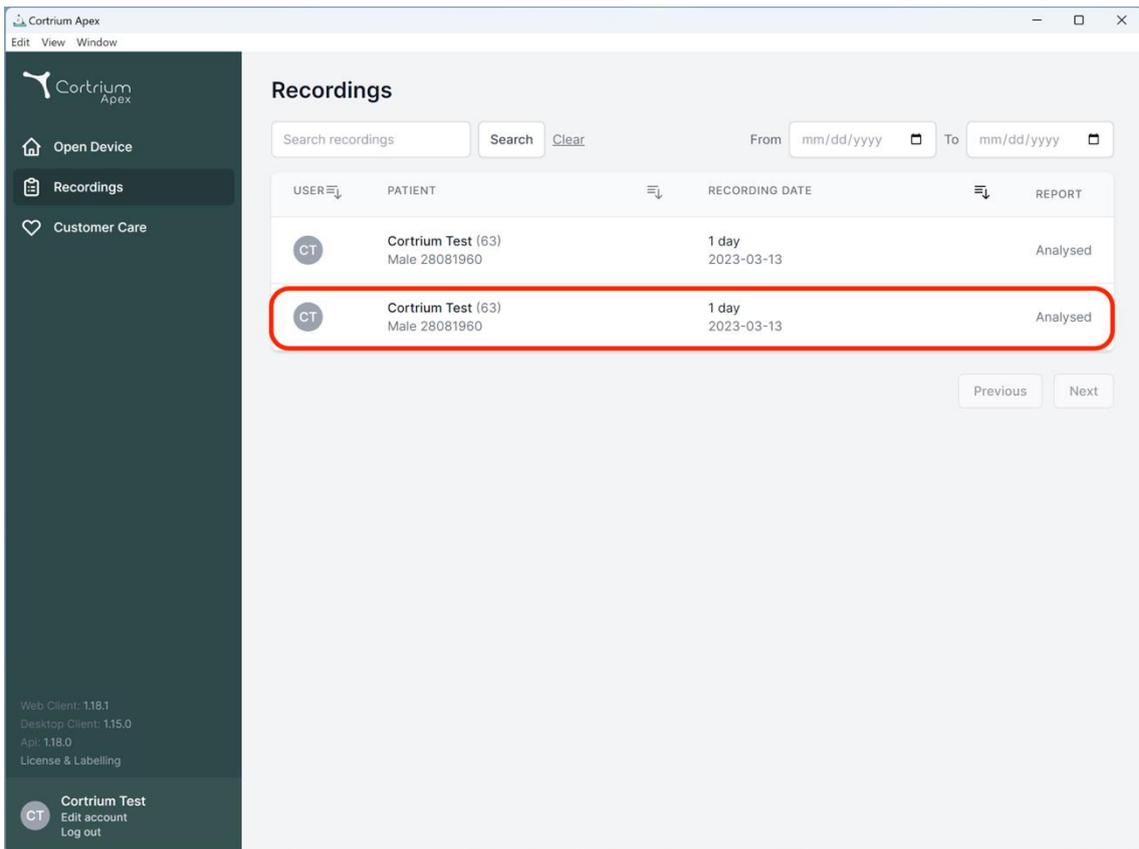
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. The interface includes a dark sidebar on the left with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area features a search bar, date filters, and a table of recordings. The table has the following structure:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

The second row of the table is highlighted with a red border. Below the table are 'Previous' and 'Next' navigation buttons. The sidebar at the bottom shows the user 'Cortrium Test' with options to 'Edit account' and 'Log out'. Version information is also present: Web Client: 1.18.1, Desktop Client: 1.15.0, and Api: 1.18.0.

Cortrium Apex  
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1  
Desktop Client: 1.15.0  
App: 1.18.0  
License & Labelling

**CT** Cortrium Test  
Edit account  
Log out

### Recording details

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

#### Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

#### Recording Summary

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

#### Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
 HRV  AV-block  Fast Track

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
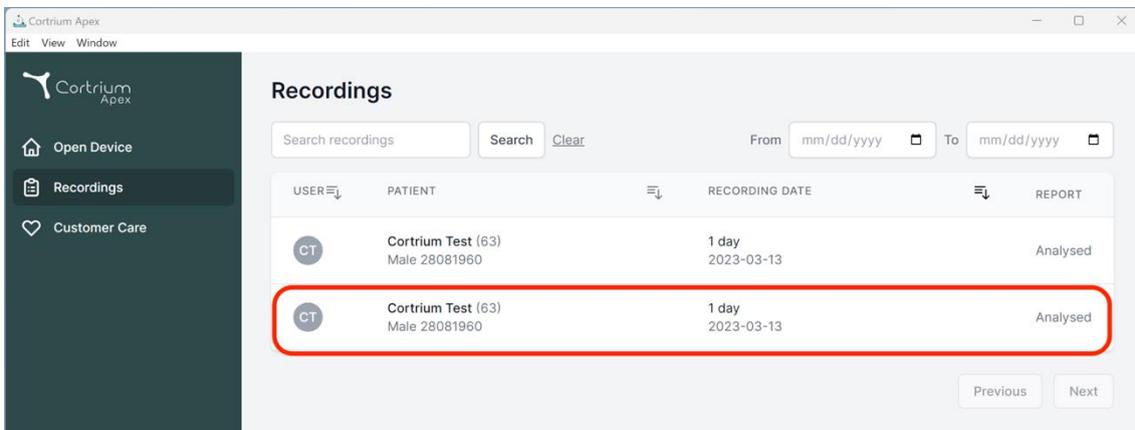
The screenshot shows the 'Recordings' section of the Cortrium Apex software. It features a search bar, a date range selector, and a table with columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row of the table is highlighted with a red box.

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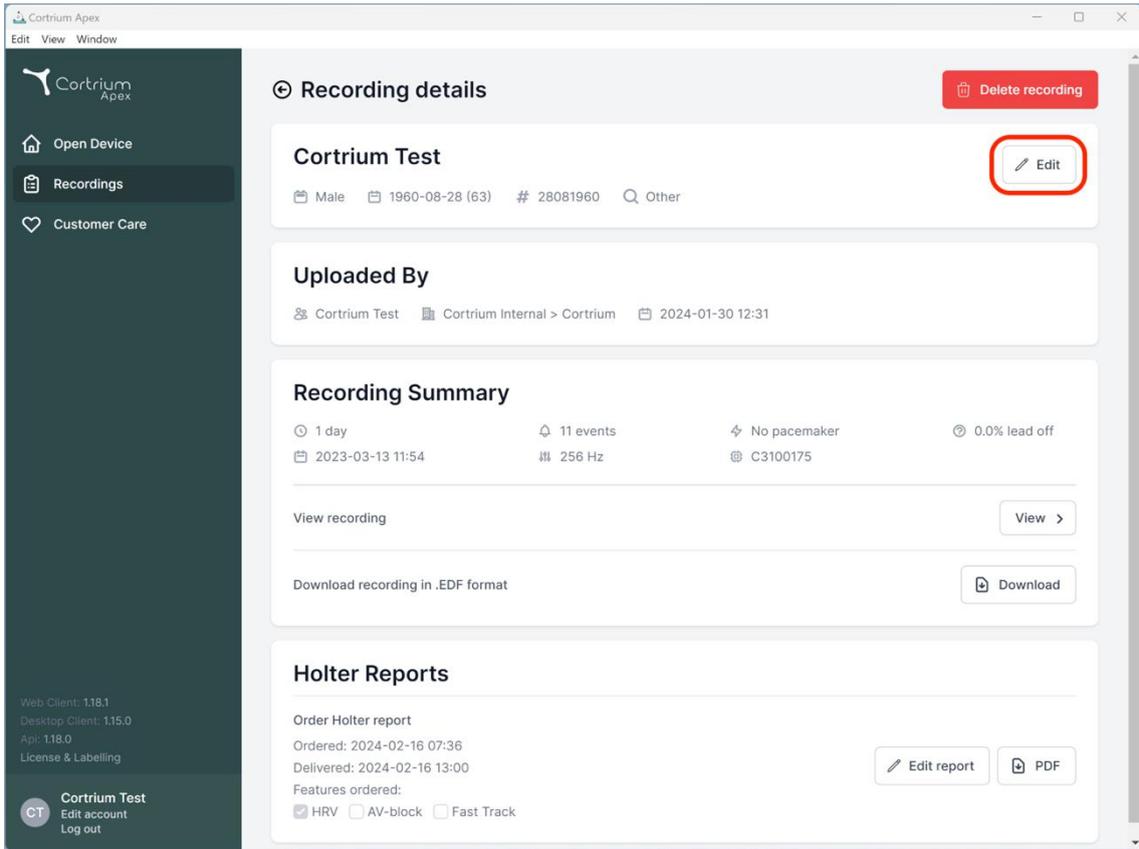**Recording details** Delete recording

**Cortrium Test** Edit

Male | 1960-08-28 (63) | # 28081960 | Other

---

**Uploaded By**

Cortrium Test | Cortrium Internal > Cortrium | 2024-01-30 12:31

---

**Recording Summary**

1 day | 11 events | No pacemaker | 0.0% lead off  
 2023-03-13 11:54 | 256 Hz | C3100175

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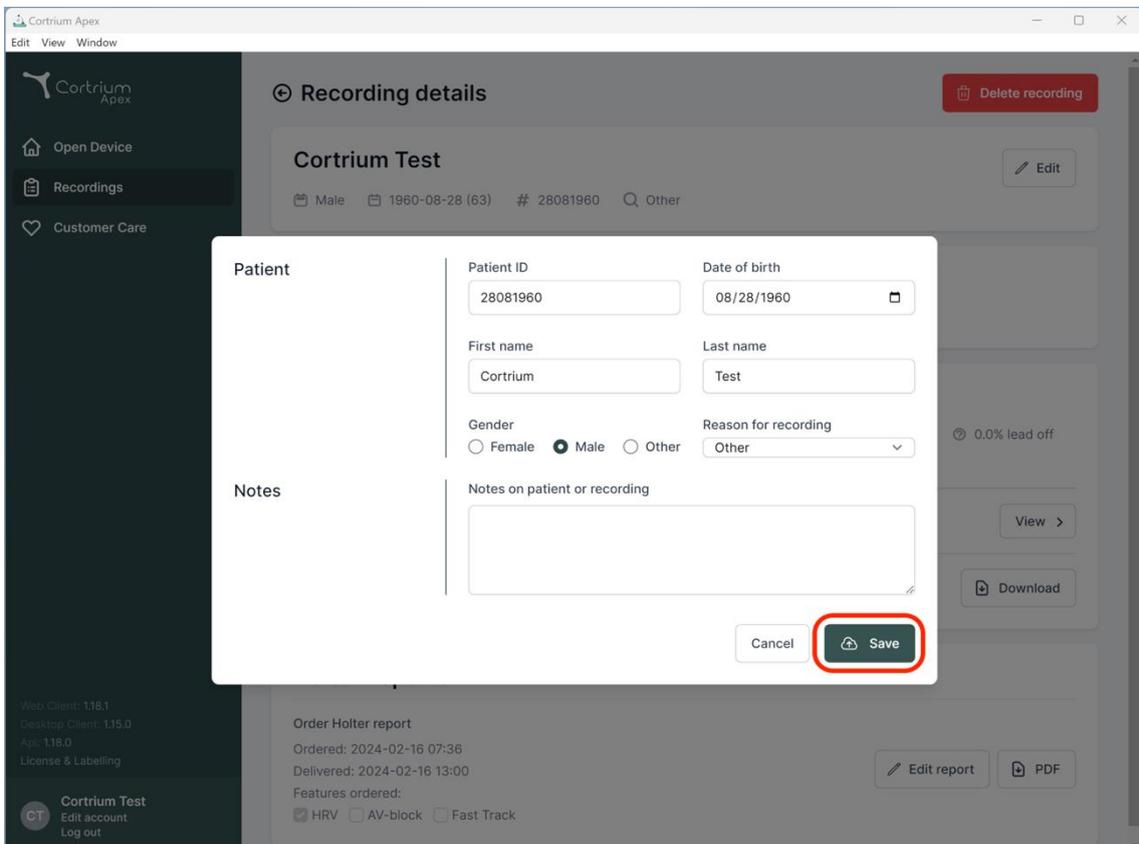
Patient ID: 28081960 | Date of birth: 08/28/1960

First name: Cortrium | Last name: Test

Gender:  Female  Male  Other | Reason for recording: Other

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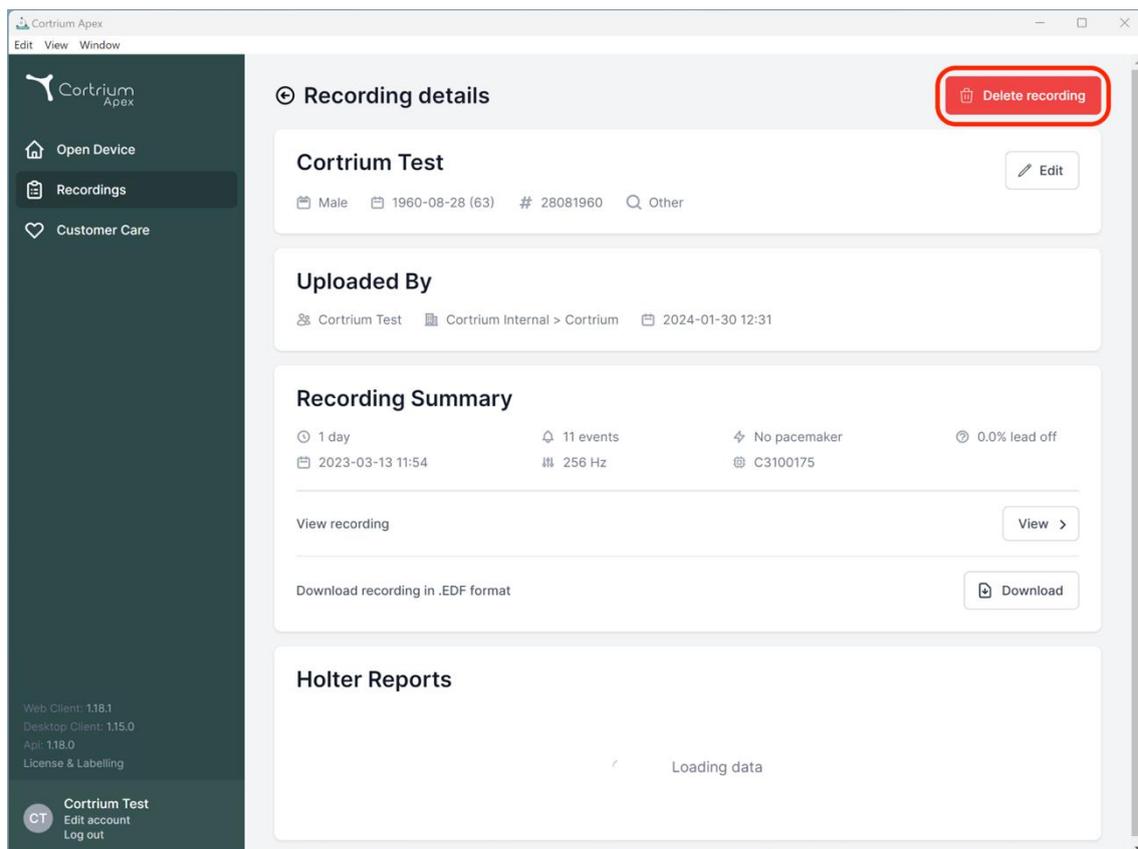
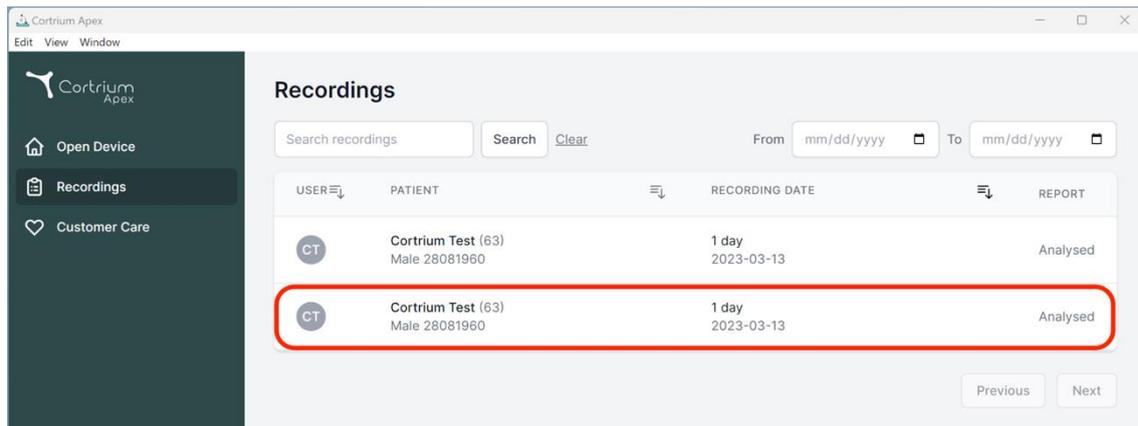
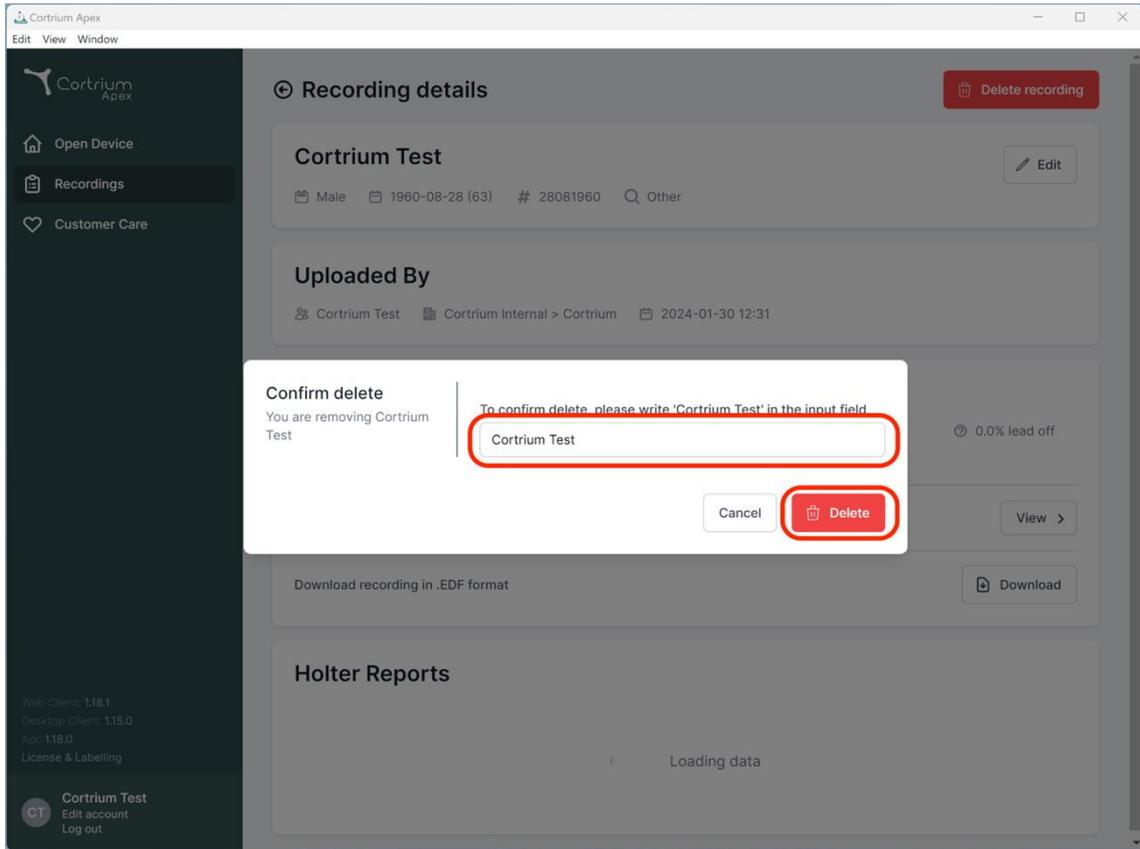
The screenshot shows the Cortrium Apex web application interface. A 'Confirm delete' dialog box is open in the foreground, overlaid on the 'Recording details' page. The dialog box contains the following text and elements:

- Confirm delete**
- You are removing Cortrium Test
- To confirm delete, please write 'Cortrium Test' in the input field
- An input field containing the text 'Cortrium Test' (highlighted with a red circle).
- Buttons for 'Cancel' and 'Delete' (the 'Delete' button is highlighted with a red circle).

The background page, 'Recording details', shows the following information:

- Recording details** (with a 'Delete recording' button)
- Cortrium Test** (with an 'Edit' button)
- Metadata: Male, 1960-08-28 (63), # 28081960, Other
- Uploaded By**: Cortrium Test, Cortrium Internal > Cortrium, 2024-01-30 12:31
- 0.0% lead off
- Download recording in .EDF format (with a 'Download' button)
- Holter Reports**: Loading data

The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The bottom left corner shows user information for 'Cortrium Test' (CT) with options to 'Edit account' and 'Log out'.

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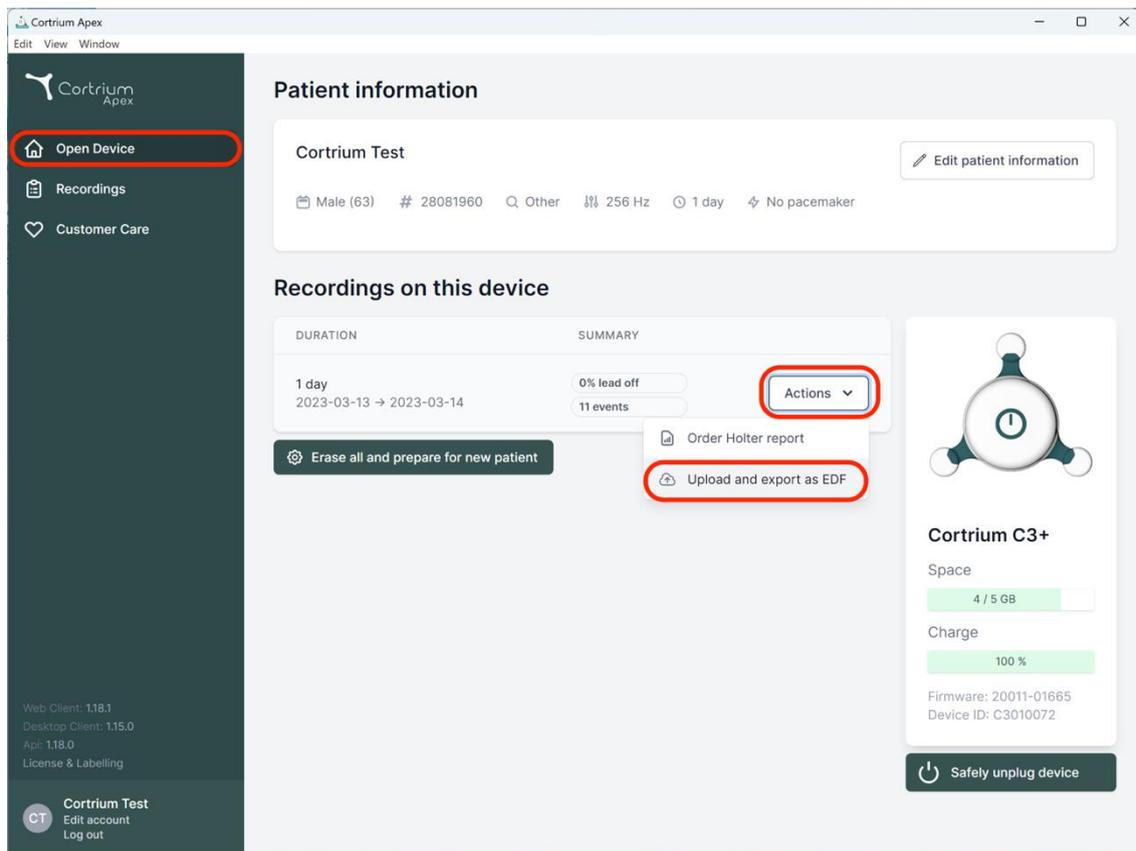
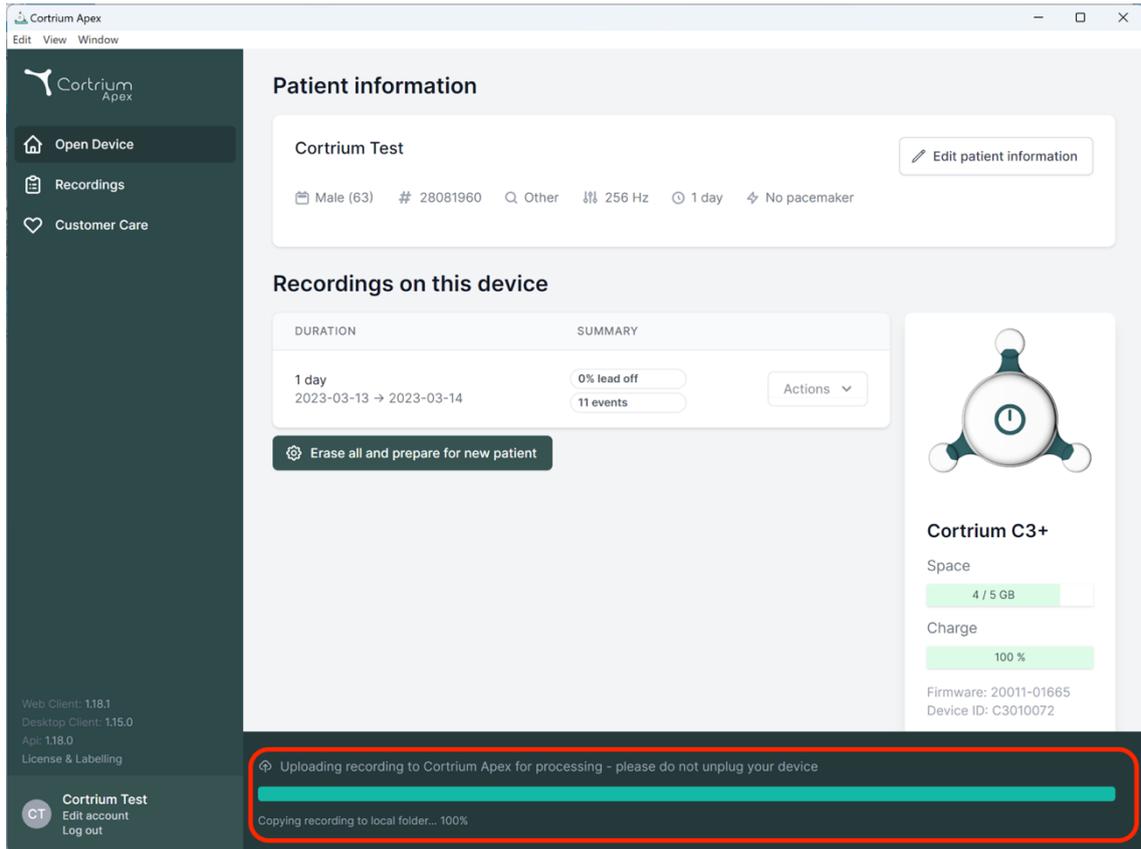
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">View &gt;</a>

[Erase all and prepare for new patient](#)

**Cortrium C3+**

Space: 4 / 5 GB

Charge: 100 %

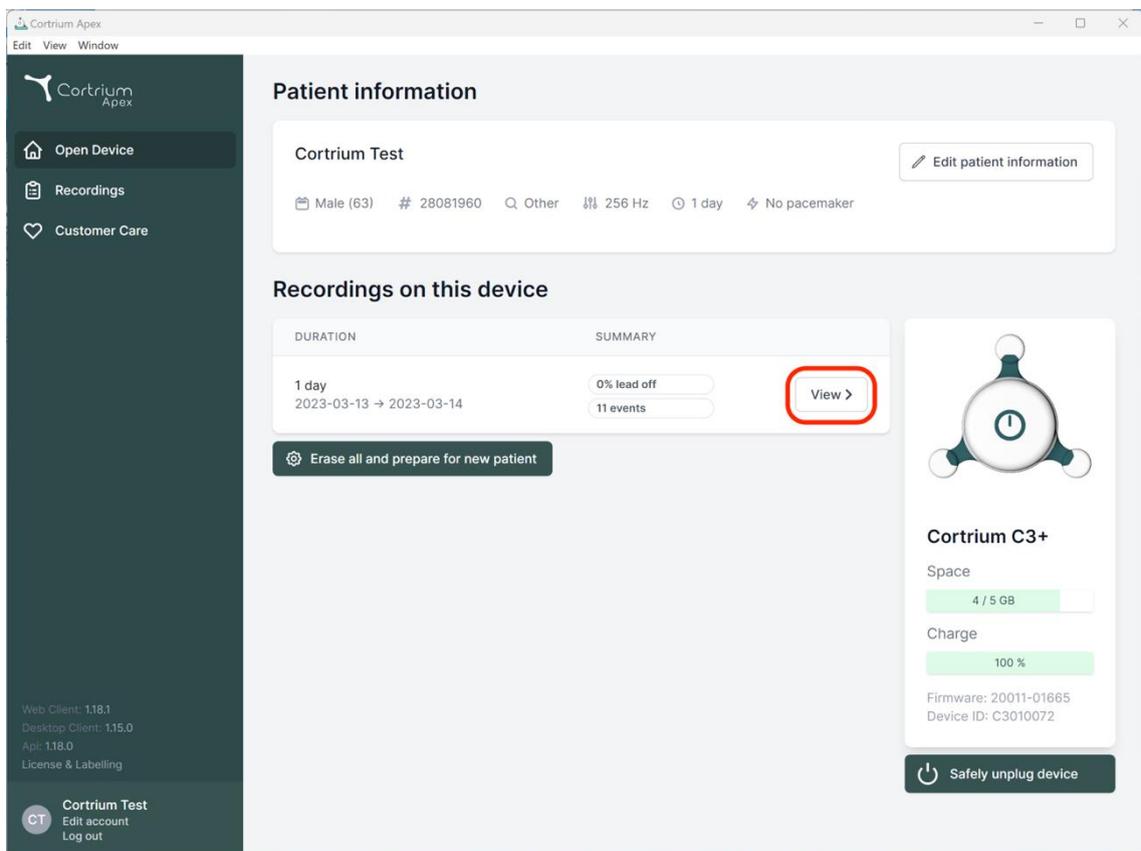
Firmware: 20011-01665  
Device ID: C3010072

Web Client: 1.18.1  
Desktop Client: 1.15.0  
App: 1.18.0  
License & Labelling

**Cortrium Test**  
CT Edit account Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

**Cortrium Apex**  
Edit View Window

**Patient information**

**Cortrium Test** [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">View &gt;</a>

[Erase all and prepare for new patient](#)

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
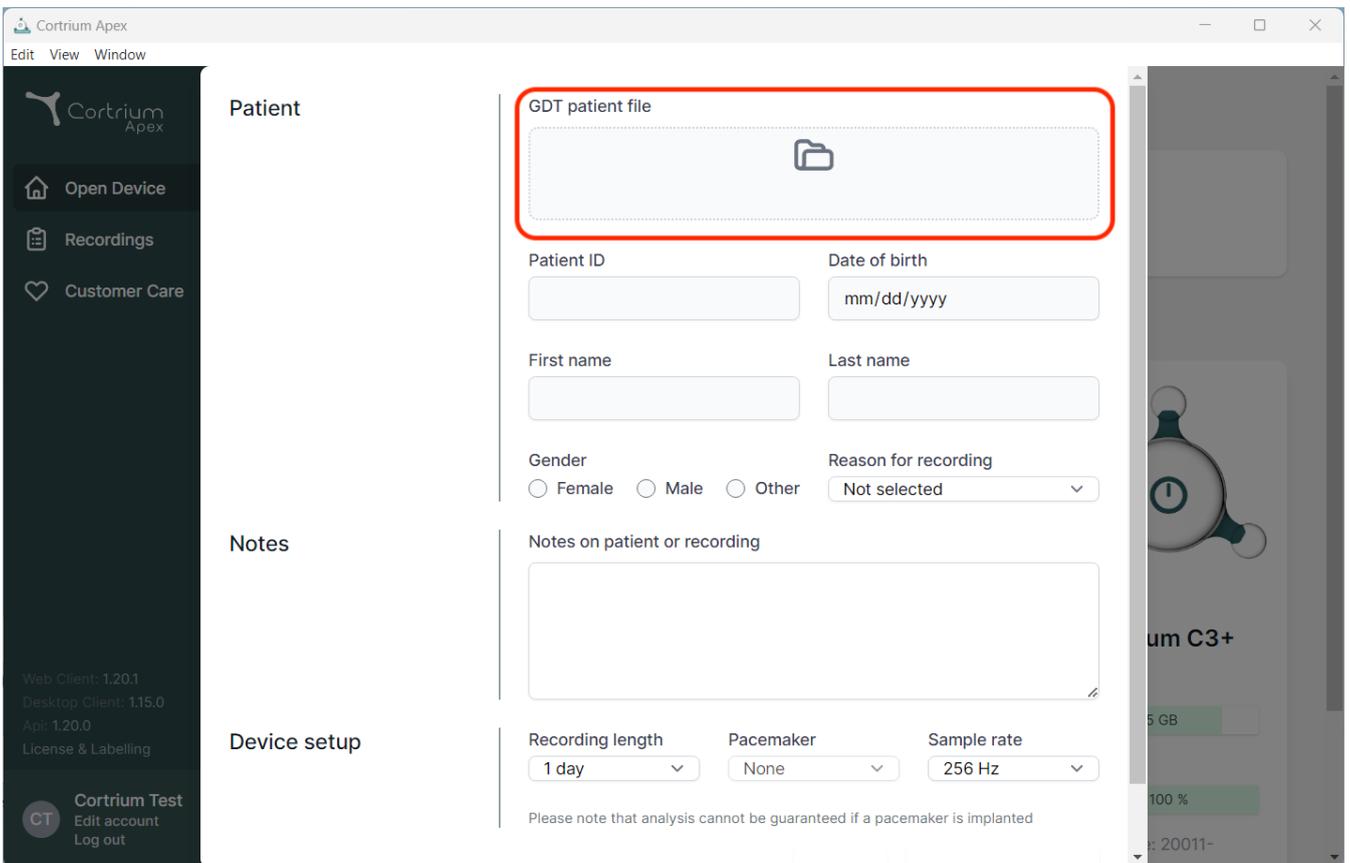
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red box highlights the "GDT patient file" field, which is a file upload area. Below this field are several input fields: "Patient ID", "Date of birth" (with a placeholder "mm/dd/yyyy"), "First name", "Last name", "Gender" (with radio buttons for Female, Male, and Other), and "Reason for recording" (a dropdown menu currently set to "Not selected"). Below these fields is a "Notes" section with a text area labeled "Notes on patient or recording". At the bottom of the form is the "Device setup" section, which includes "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below the device setup section states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". The bottom left corner shows the user's name "Cortrium Test" and options to "Edit account" and "Log out". The top left corner shows the application name "Cortrium Apex" and menu options "Edit", "View", and "Window".

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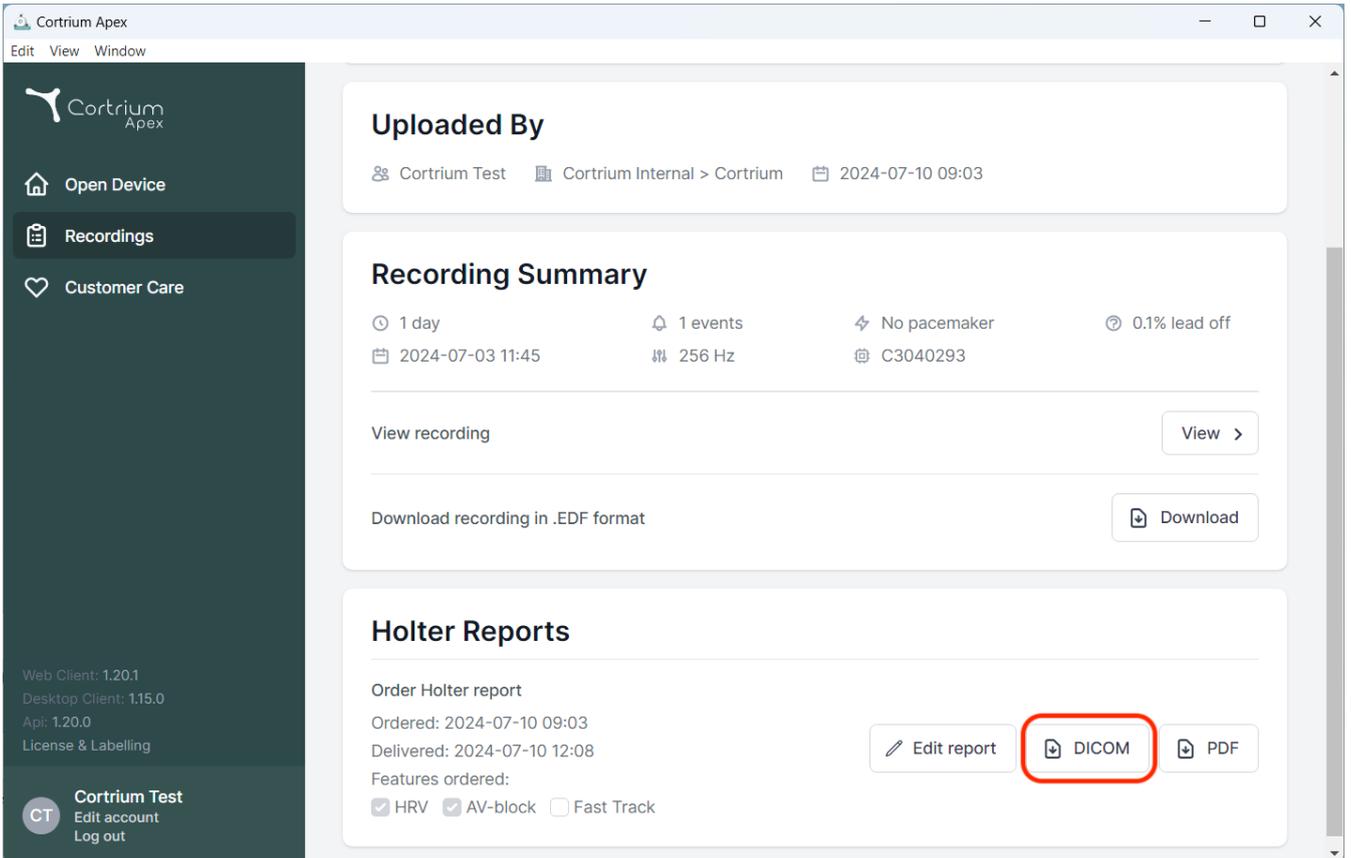
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' for 'Cortrium Internal > Cortrium' on '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' (2024-07-03 11:45), '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off' (C3040293). It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with 'CT' initials and options to 'Edit account' and 'Log out'. Version information (Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0) and 'License & Labelling' are also visible.

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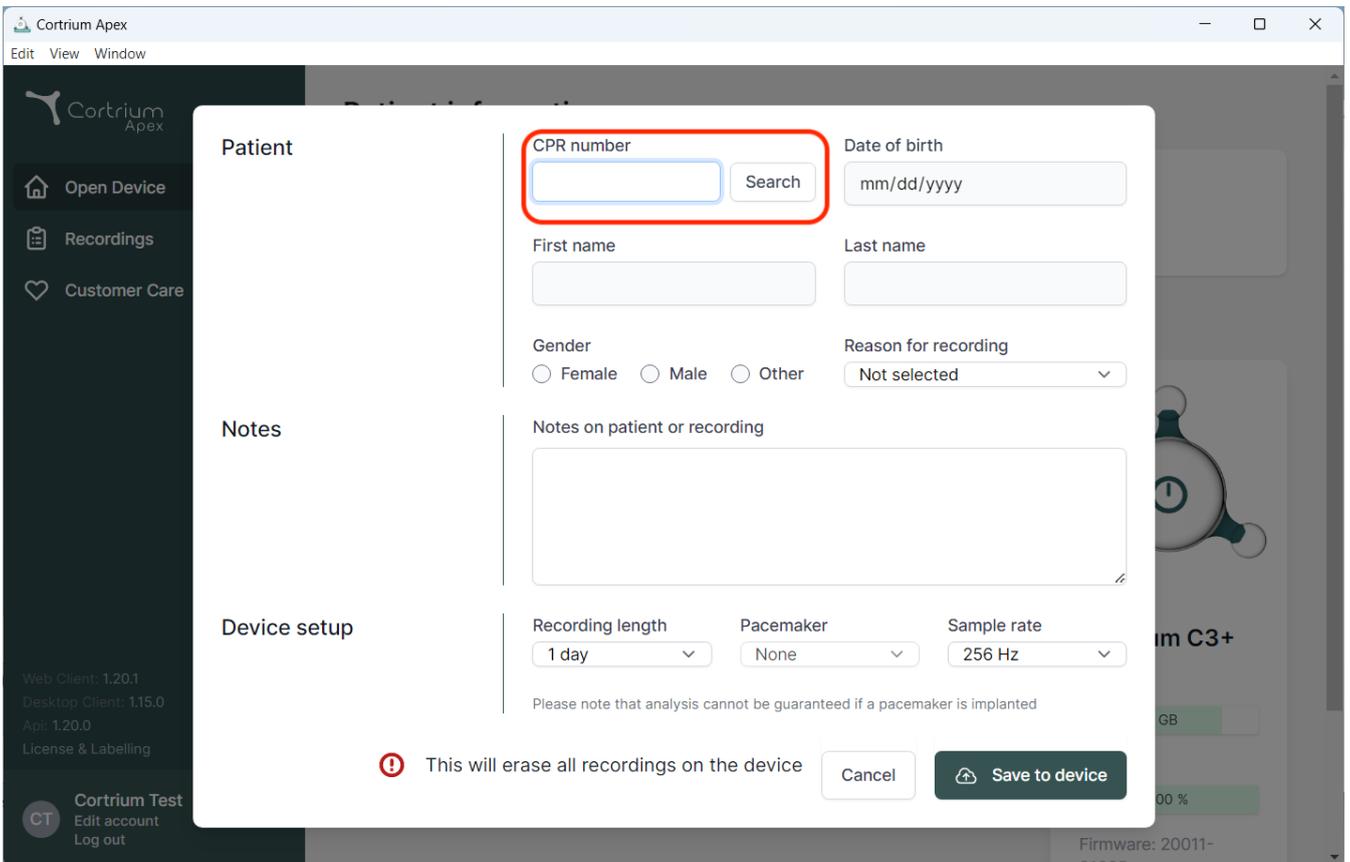Recordings

Customer Care

Web Client: 1.20.1  
Desktop Client: 1.15.0  
App: 1.20.0  
License & Labelling

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**!** This will erase all recordings on the device

Cancel

09/2024

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark  
E-mail : [info@cortrium.com](mailto:info@cortrium.com) • Site Web : [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033