Cortrium Apex 1.20

Istruzioni d'uso

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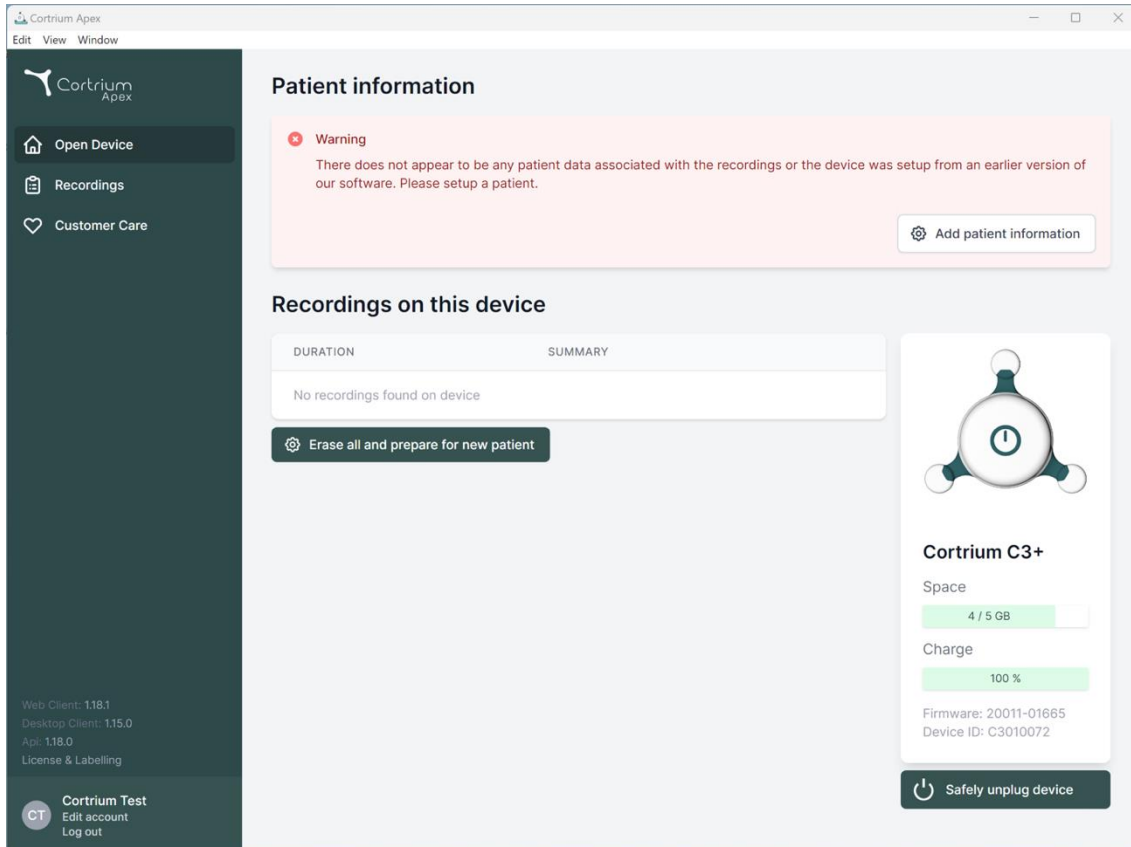
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version details (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, License & Labelling) and user options (Cortrium Test, Edit account, Log out). The main content area is titled 'Patient information' and features a warning message: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is the 'Recordings on this device' section, which contains a table with columns 'DURATION' and 'SUMMARY', currently showing 'No recordings found on device' and an 'Erase all and prepare for new patient' button. To the right is a device status card for 'Cortrium C3+' showing 'Space' at 4 / 5 GB, 'Charge' at 100%, and 'Firmware: 20011-01665 Device ID: C3010072'. A 'Safely unplug device' button is located at the bottom of the card.

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Cortrium Apex
Edit View Window

Cortrium Apex

Open Device
Recordings
Customer Care

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: Female Male Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

⚠ This will erase all recordings on the device

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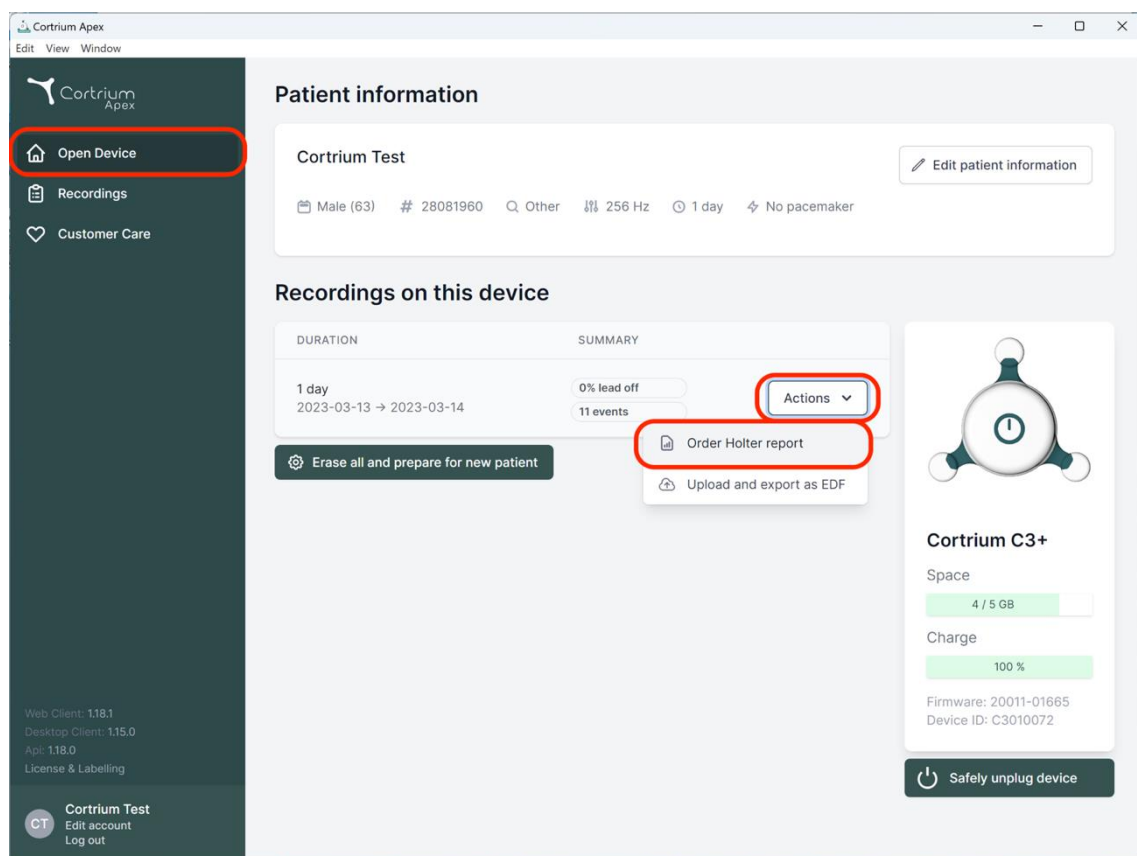
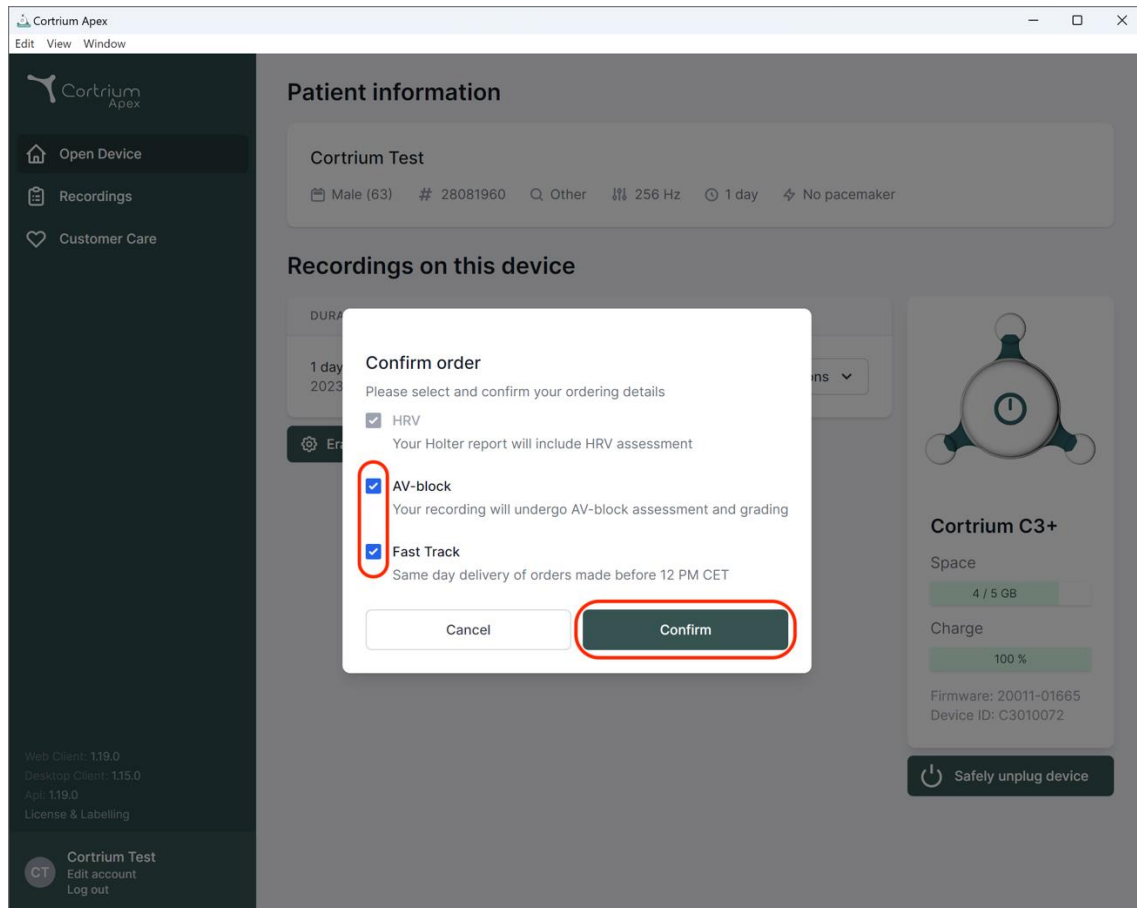
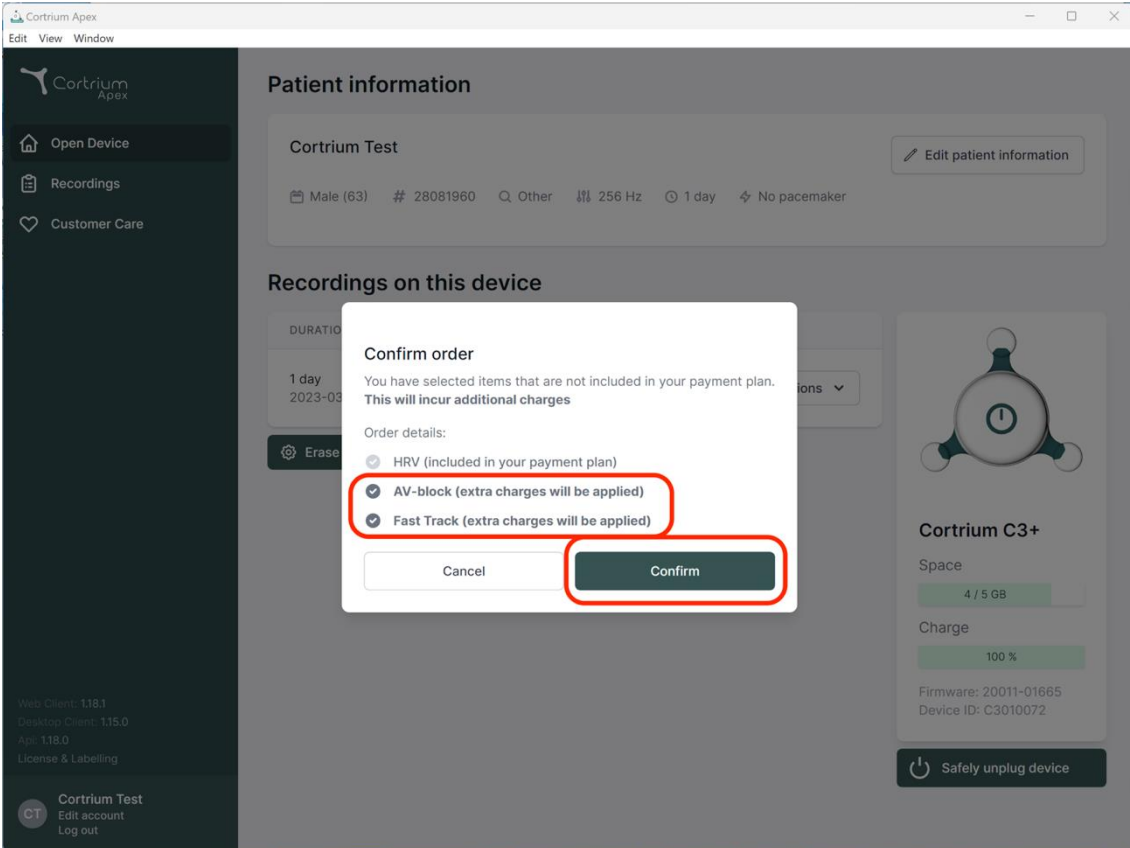
Erase

Confirm order

You have selected items that are not included in your payment plan. **This will incur additional charges**

Order details:

- HRV (included in your payment plan)
- AV-block (extra charges will be applied)**
- Fast Track (extra charges will be applied)**

Cancel Confirm

Cortrium C3+

Space

4 / 5 GB

Charge

100 %

Firmware: 20011-01665


Device ID: C3010072

Safely unplug device

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
CT Edit account
Log out

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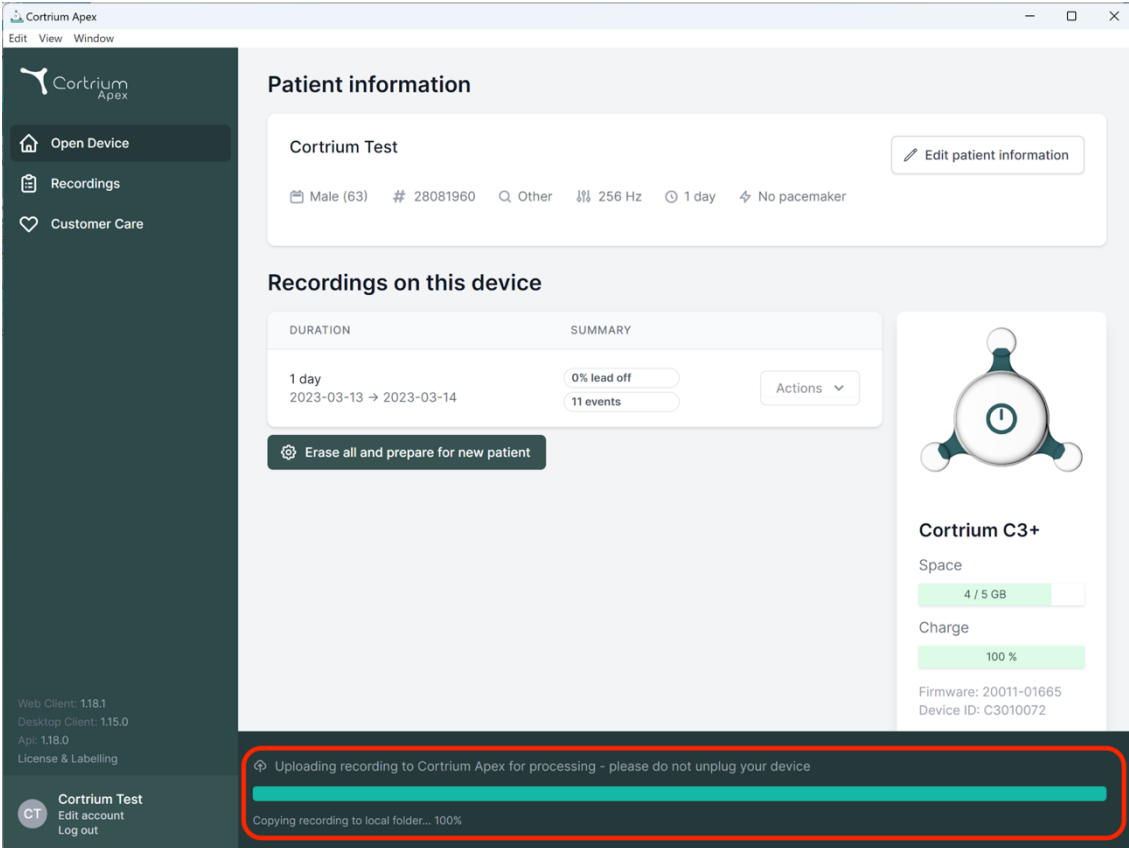The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and device settings (256 Hz, 1 day, No pacemaker). Below this is a 'Recordings on this device' section with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A 'Erase all and prepare for new patient' button is visible. To the right, a device status card for 'Cortrium C3+' shows 4/5 GB space and 100% charge. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. The sidebar also contains version information and user options for 'Cortrium Test'.

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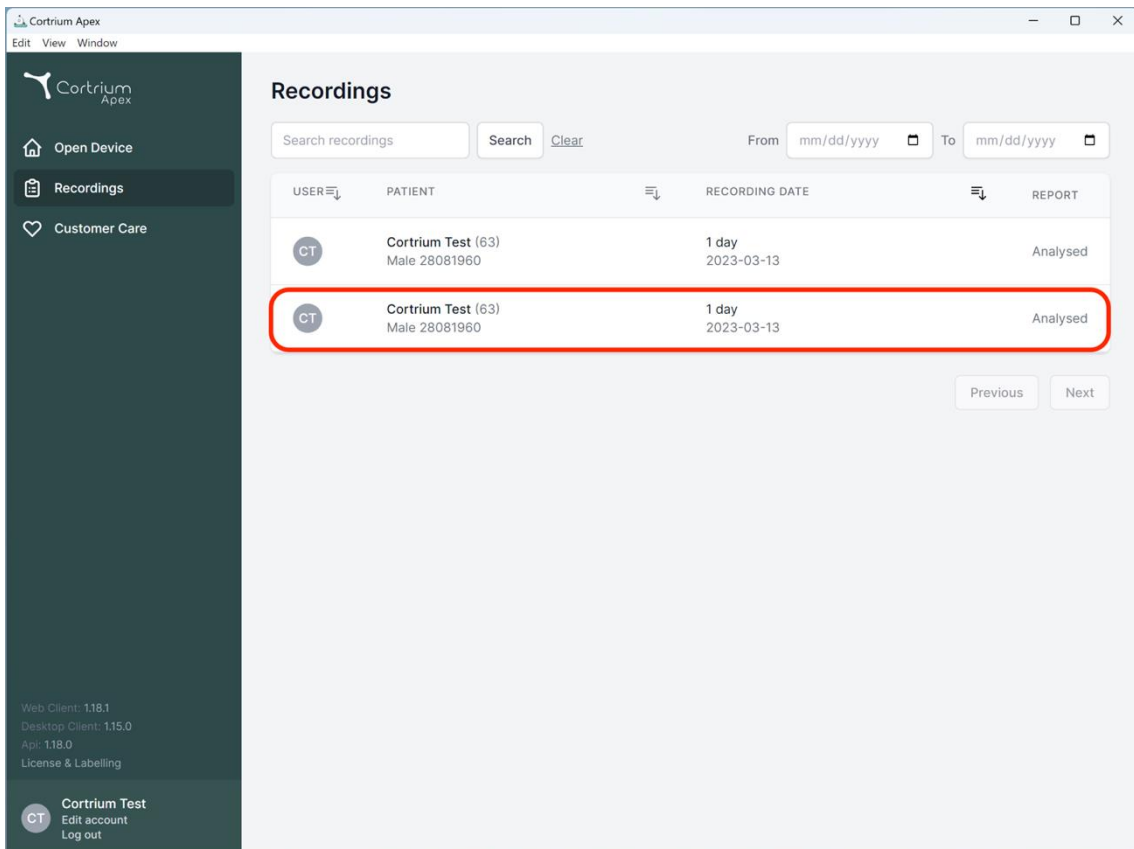
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. The interface features a dark sidebar on the left with navigation options: 'Open Device', 'Recordings' (selected), and 'Customer Care'. The main content area shows a table of recordings with the following data:

| USER | PATIENT | RECORDING DATE | REPORT |
|------|-------------------------------------|---------------------|----------|
| CT | Cortrium Test (63) Male 28081960 | 1 day 2023-03-13 | Analysed |
| CT | Cortrium Test (63) Male 28081960 | 1 day 2023-03-13 | Analysed |

Below the table are 'Previous' and 'Next' navigation buttons. The bottom of the sidebar contains version information (Web Client: 1.18.1, Desktop Client: 1.15.0, Api: 1.18.0, License & Labelling) and a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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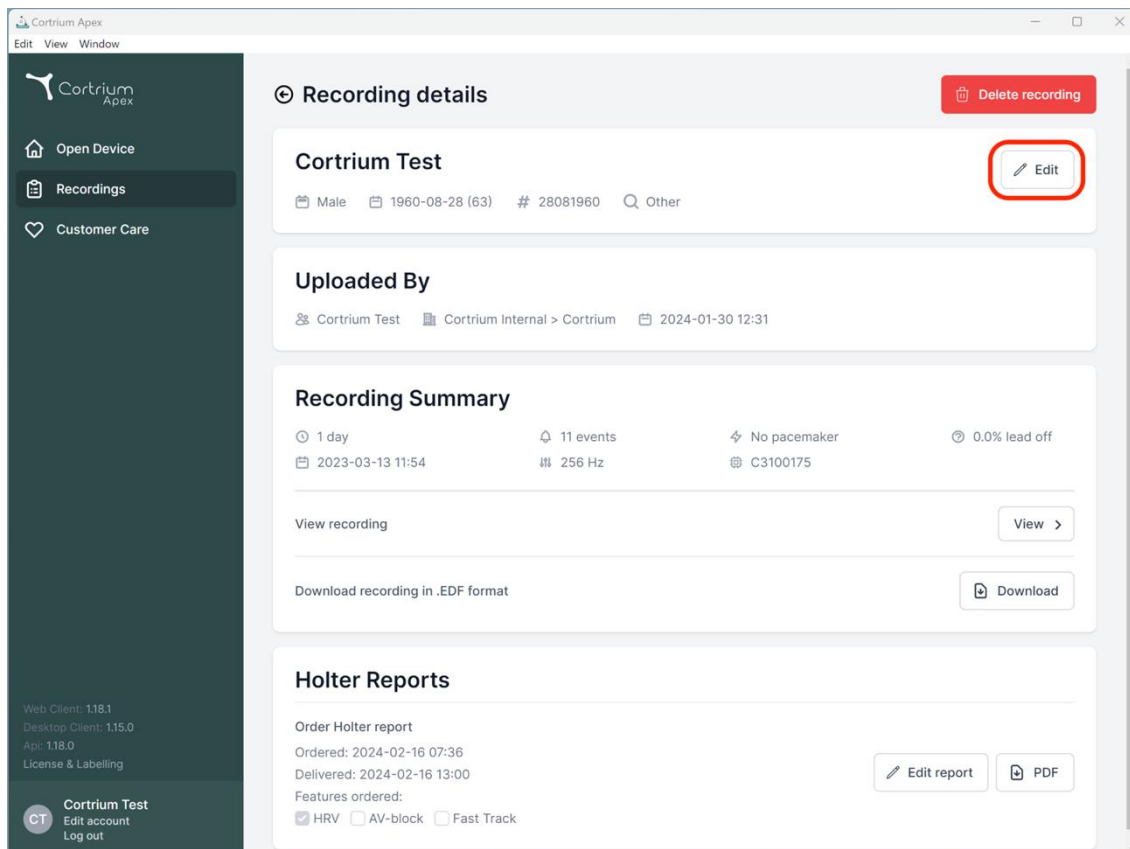
Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

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Holter Reports

Order Holter report
 Ordered: 2024-02-16 07:36
 Delivered: 2024-02-16 13:00
 Features ordered:
 HRV AV-block Fast Track Edit report PDF

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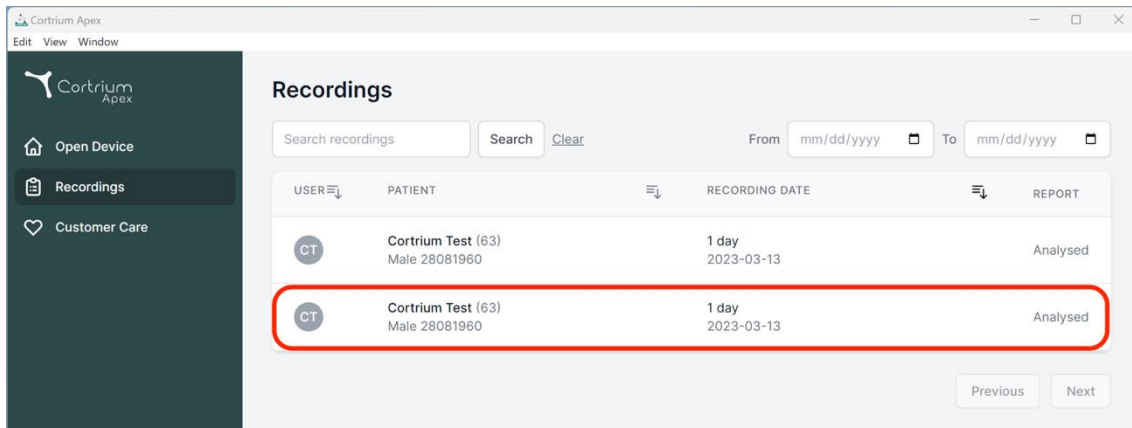
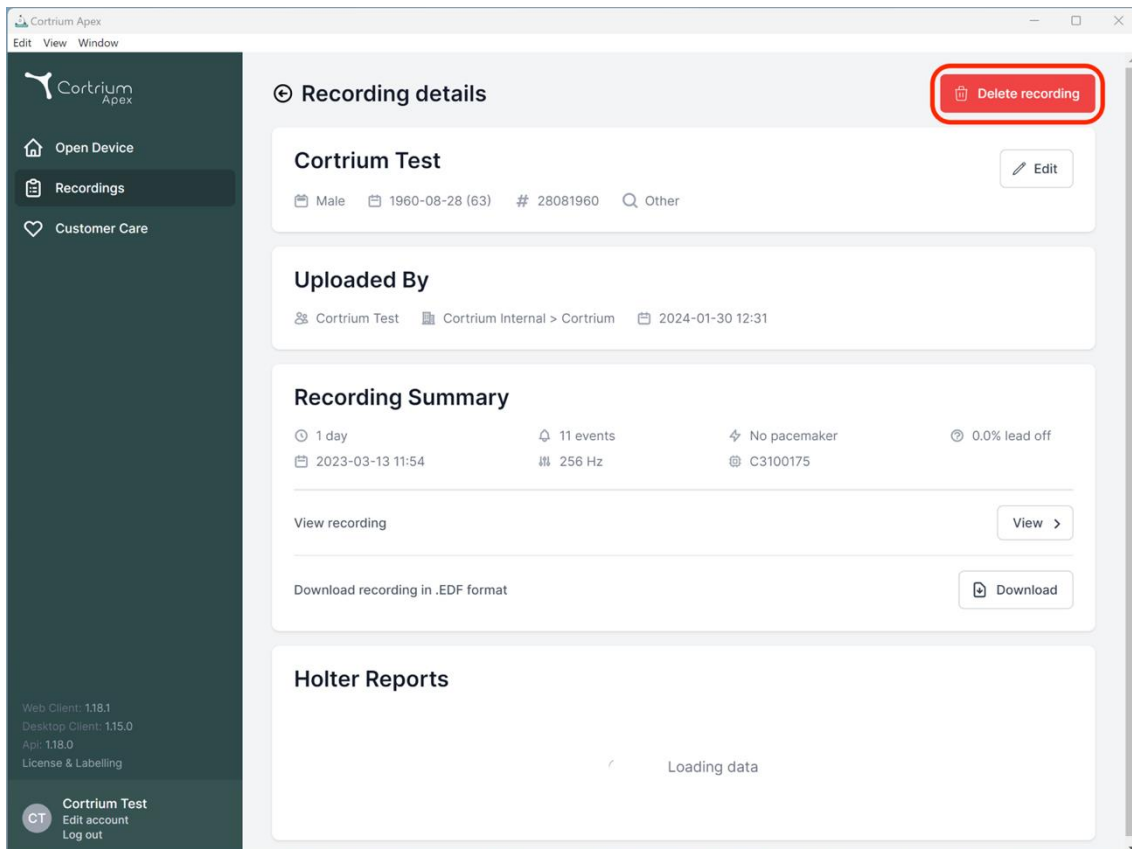
Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

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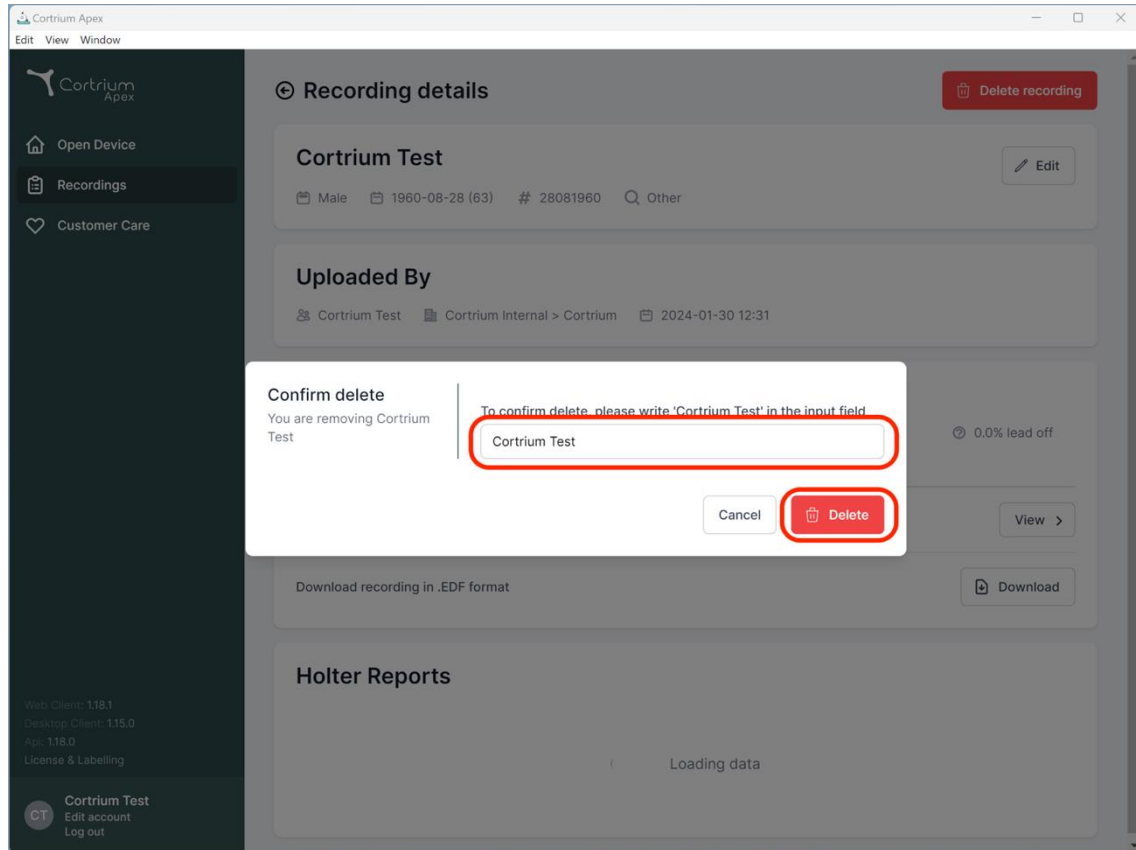
Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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
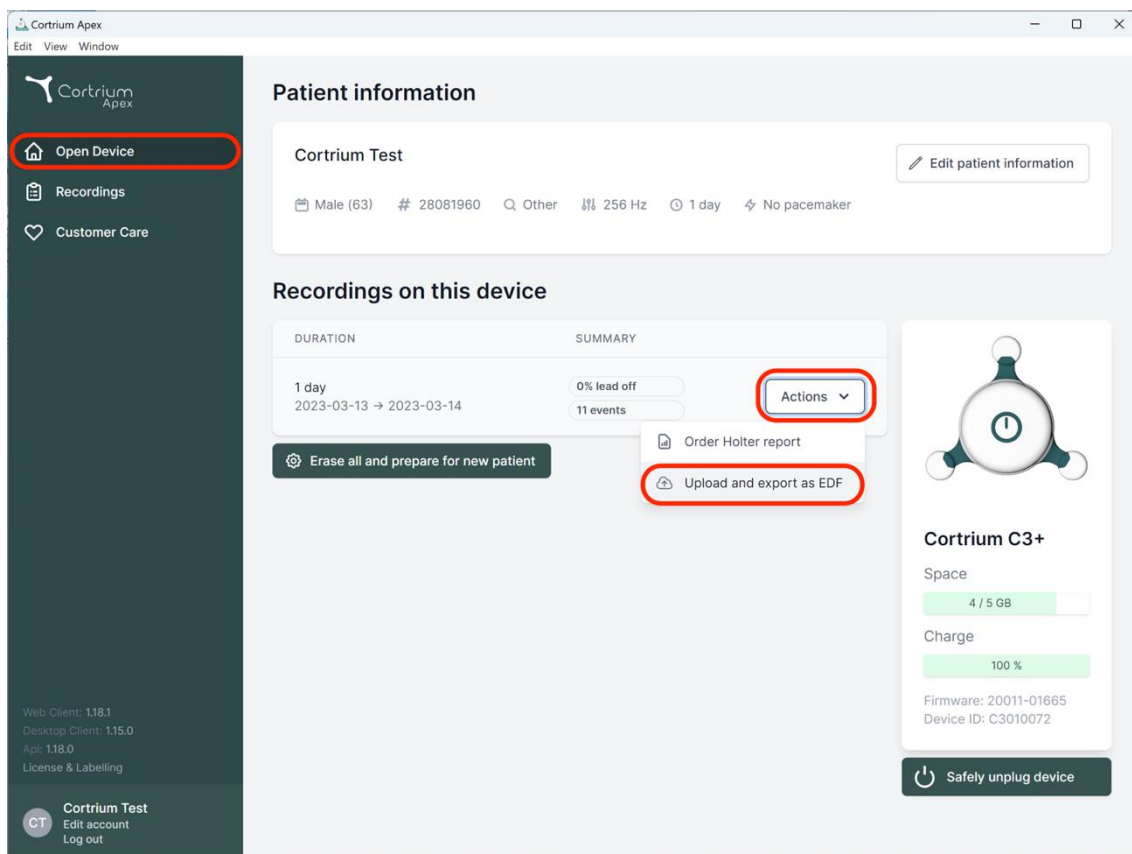
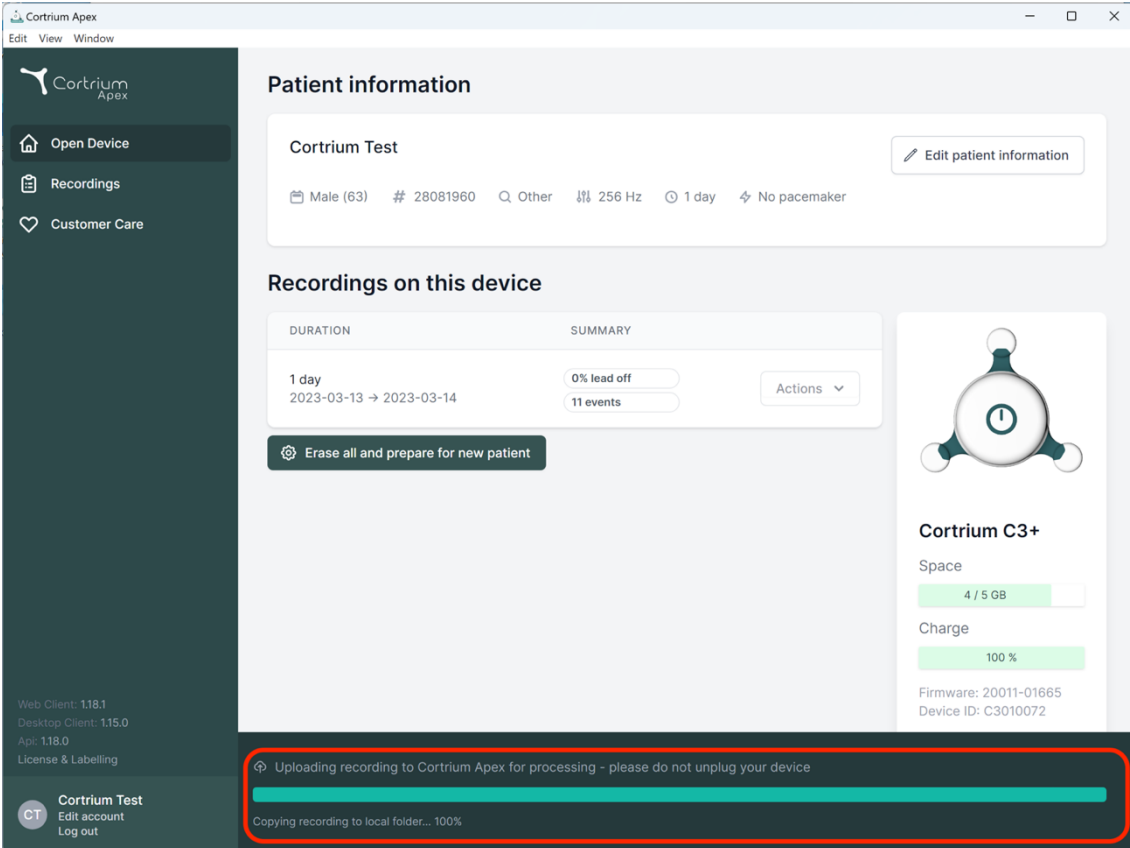
The screenshot shows the Cortrium Apex web interface. The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The main content area is titled "Patient information" and displays details for "Cortrium Test", including patient demographics and device settings. Below this, the "Recordings on this device" section shows a recording for "1 day" from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A red box highlights a progress bar at the bottom of the interface, indicating the upload status: "Uploading recording to Cortrium Apex for processing - please do not unplug your device" and "Copying recording to local folder... 100%".

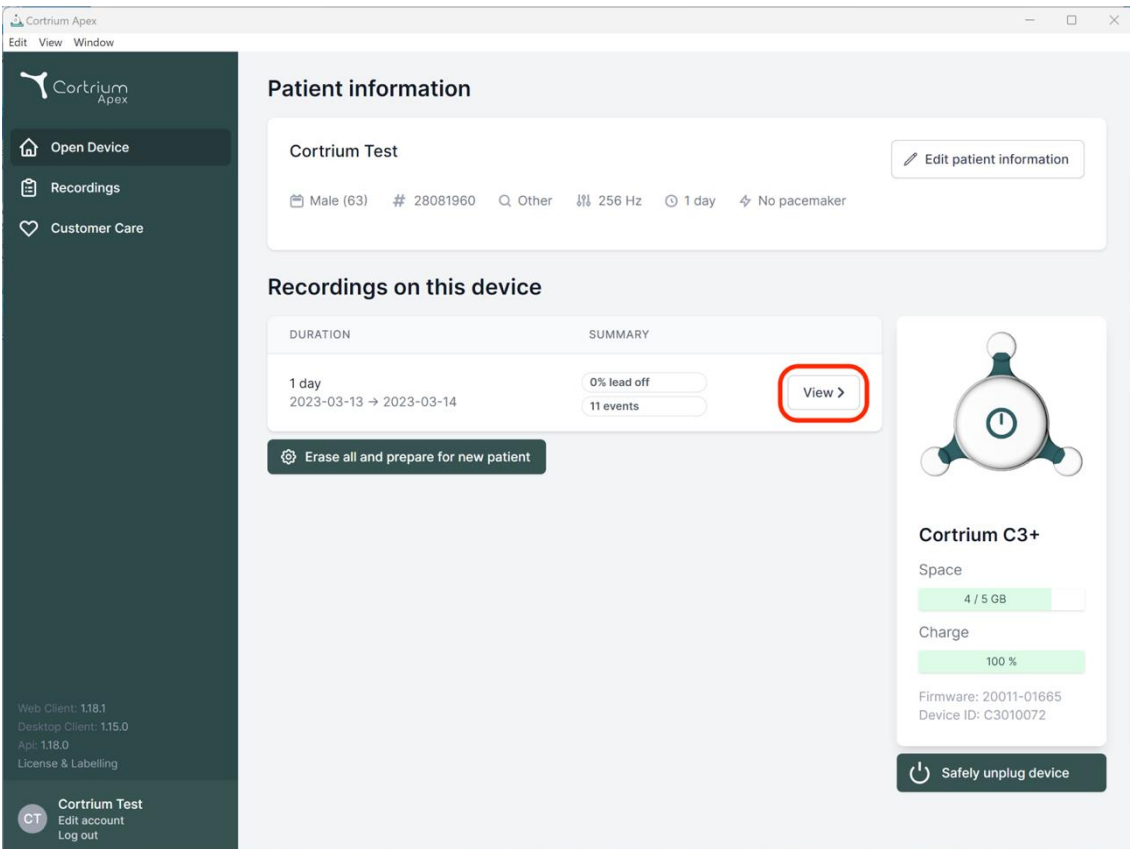The screenshot shows the same Cortrium Apex web interface. The "Recordings on this device" section is visible, showing the recording details. A red box highlights the "View >" button next to the recording summary, which is used to view the recording details.

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
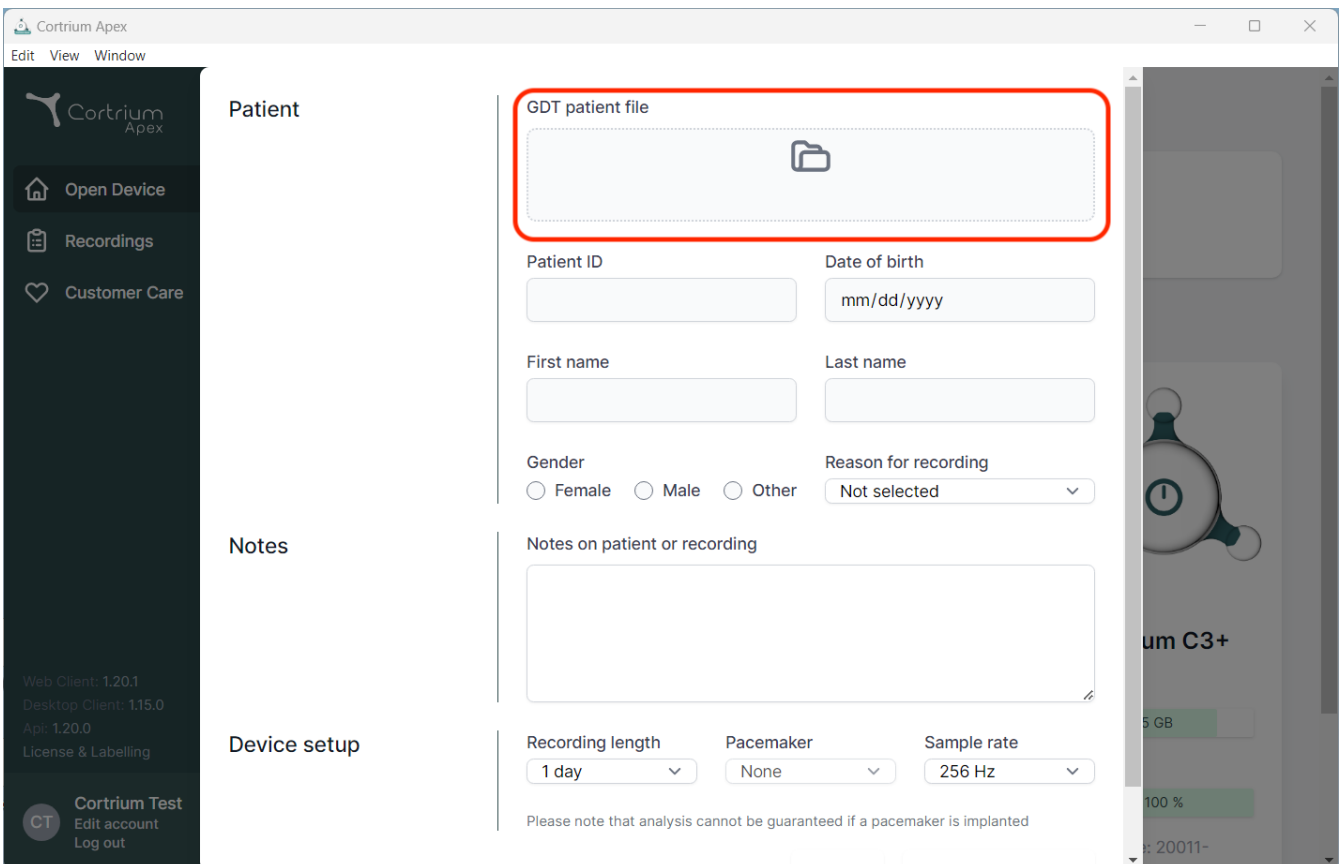
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red box highlights the "GDT patient file" input field, which is currently empty and contains a folder icon. Below this field are several input fields: "Patient ID", "Date of birth" (with a placeholder "mm/dd/yyyy"), "First name", and "Last name". There are also radio buttons for "Gender" (Female, Male, Other) and a dropdown menu for "Reason for recording" (currently set to "Not selected"). A text area labeled "Notes on patient or recording" is located below the gender and recording reason fields. At the bottom of the form, there are three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note at the bottom of the form states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". The bottom left corner shows version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". The bottom right corner shows a user profile for "Cortrium Test" with options to "Edit account" and "Log out".

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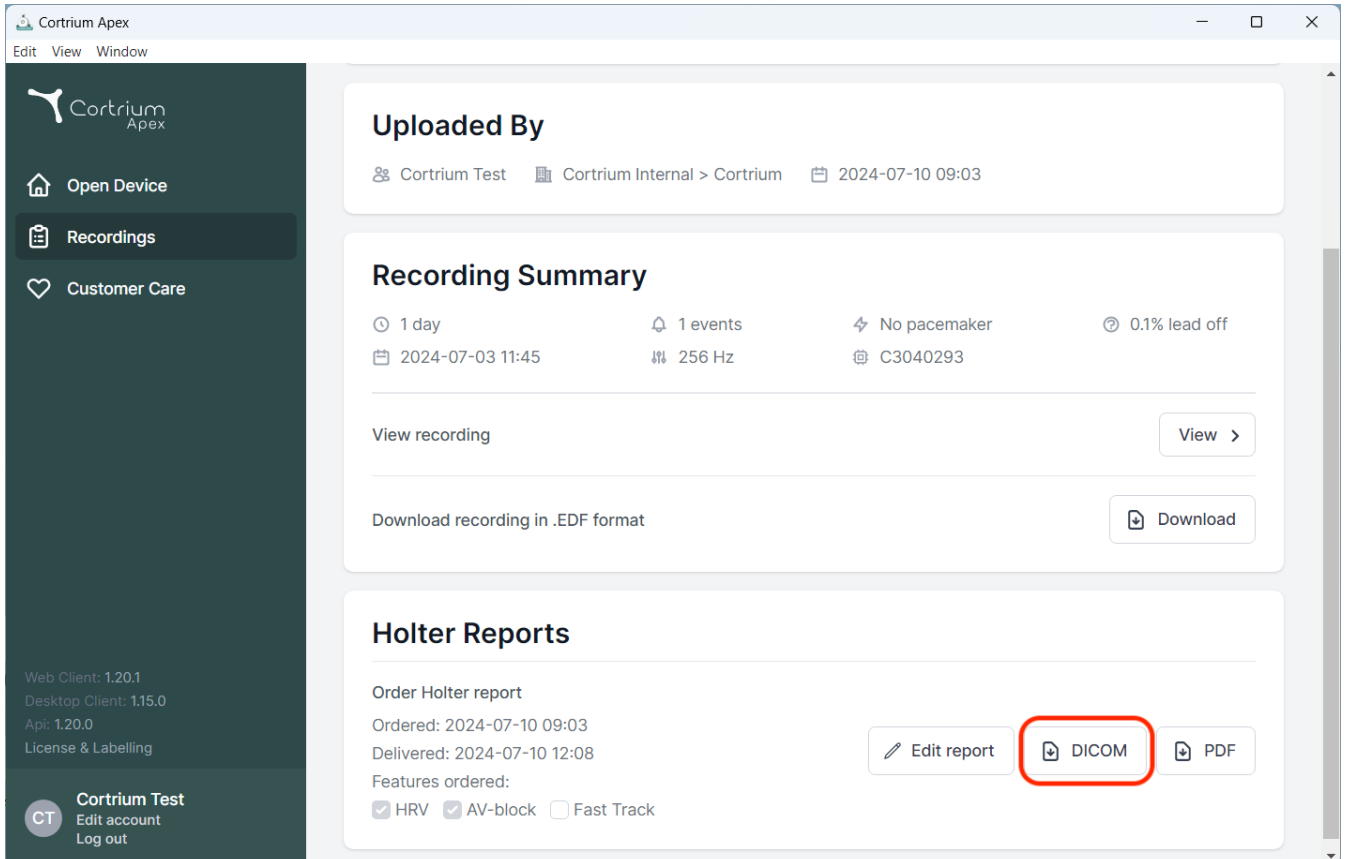
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' sampling rate, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. System version information is also visible: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', and 'Api: 1.20.0'.

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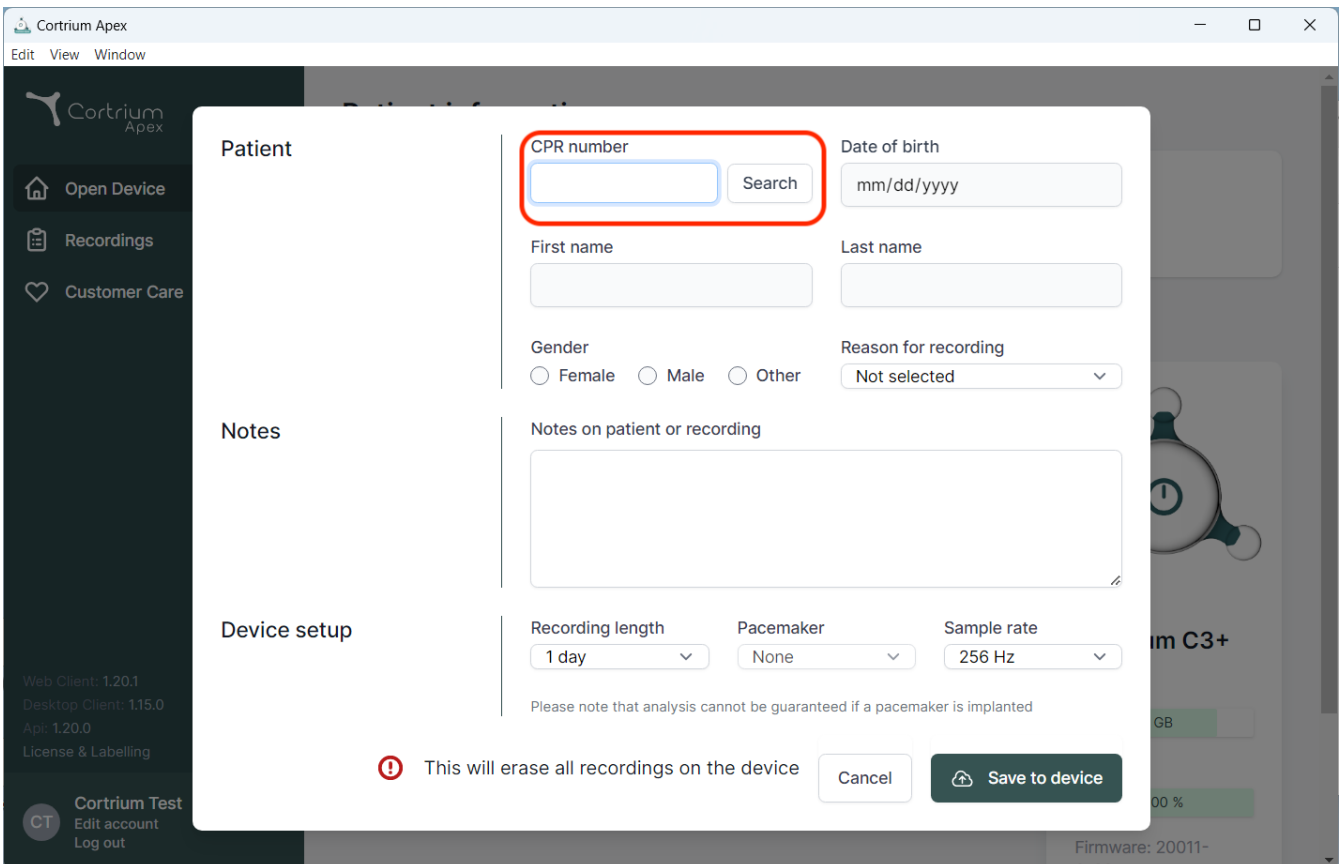The screenshot displays the 'Cortrium Apex' web application interface. A modal window titled 'Patient' is open, containing the following fields and controls:

- CPR number:** A text input field with a 'Search' button next to it, highlighted by a red rectangle.
- Date of birth:** A text input field with a placeholder 'mm/dd/yyyy'.
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for 'Female', 'Male', and 'Other'.
- Reason for recording:** A dropdown menu currently set to 'Not selected'.
- Notes:** A large text area labeled 'Notes on patient or recording'.
- Device setup:** Three dropdown menus for 'Recording length' (set to '1 day'), 'Pacemaker' (set to 'None'), and 'Sample rate' (set to '256 Hz').

Below the device setup section, a warning message is displayed: **!** This will erase all recordings on the device. At the bottom right of the modal, there are 'Cancel' and 'Save to device' buttons.

09/2024

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Denmark
Email: info@cortrium.com • Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033