Cortrium Apex 1.20

Lietošanas instrukcija

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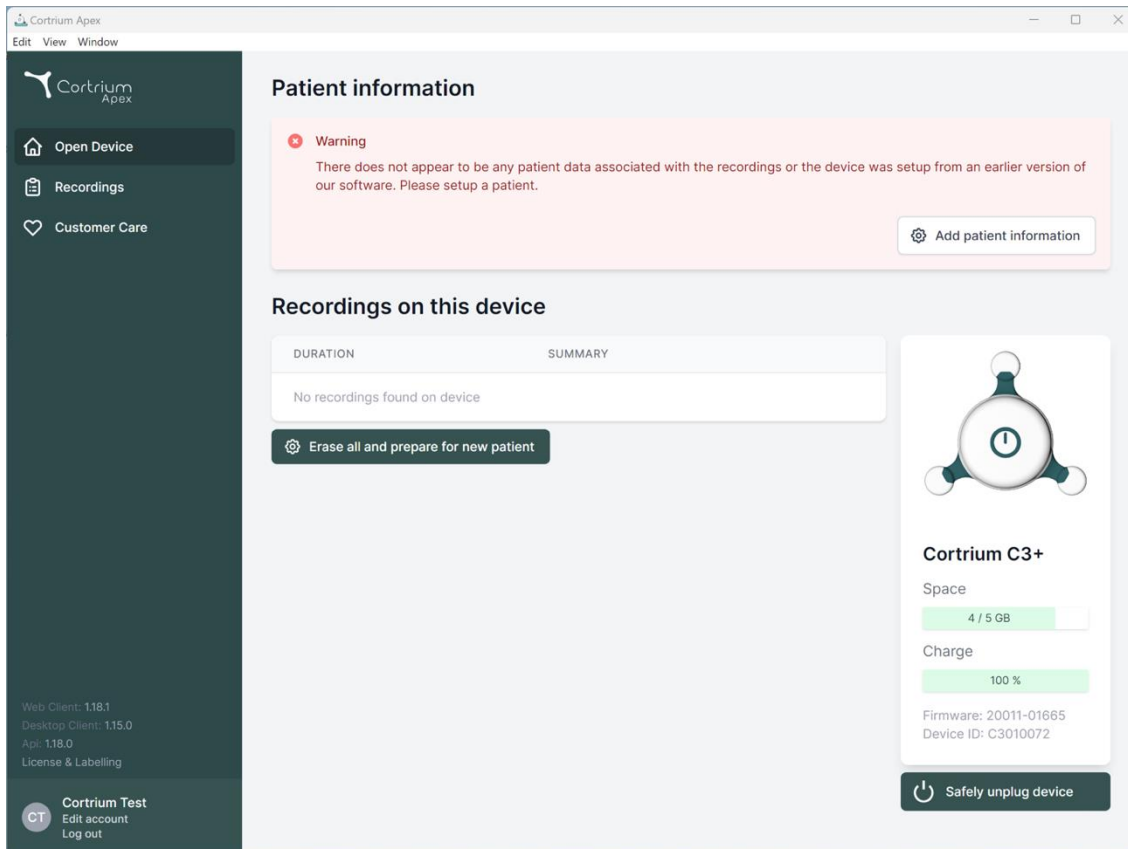
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0) and a 'License & Labelling' link. At the bottom of the sidebar is a 'Cortrium Test' section with 'Edit account' and 'Log out' options.

The main content area is titled 'Patient information' and features a pink warning box: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button.

Below this is the 'Recordings on this device' section, which contains a table with columns 'DURATION' and 'SUMMARY'. The table is empty, showing 'No recordings found on device'. A button 'Erase all and prepare for new patient' is located below the table.

On the right side, there is a device status card for 'Cortrium C3+'. It shows a battery icon, 'Space: 4 / 5 GB', 'Charge: 100%', 'Firmware: 20011-01665', and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: Female Male Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

⚠ This will erase all recordings on the device

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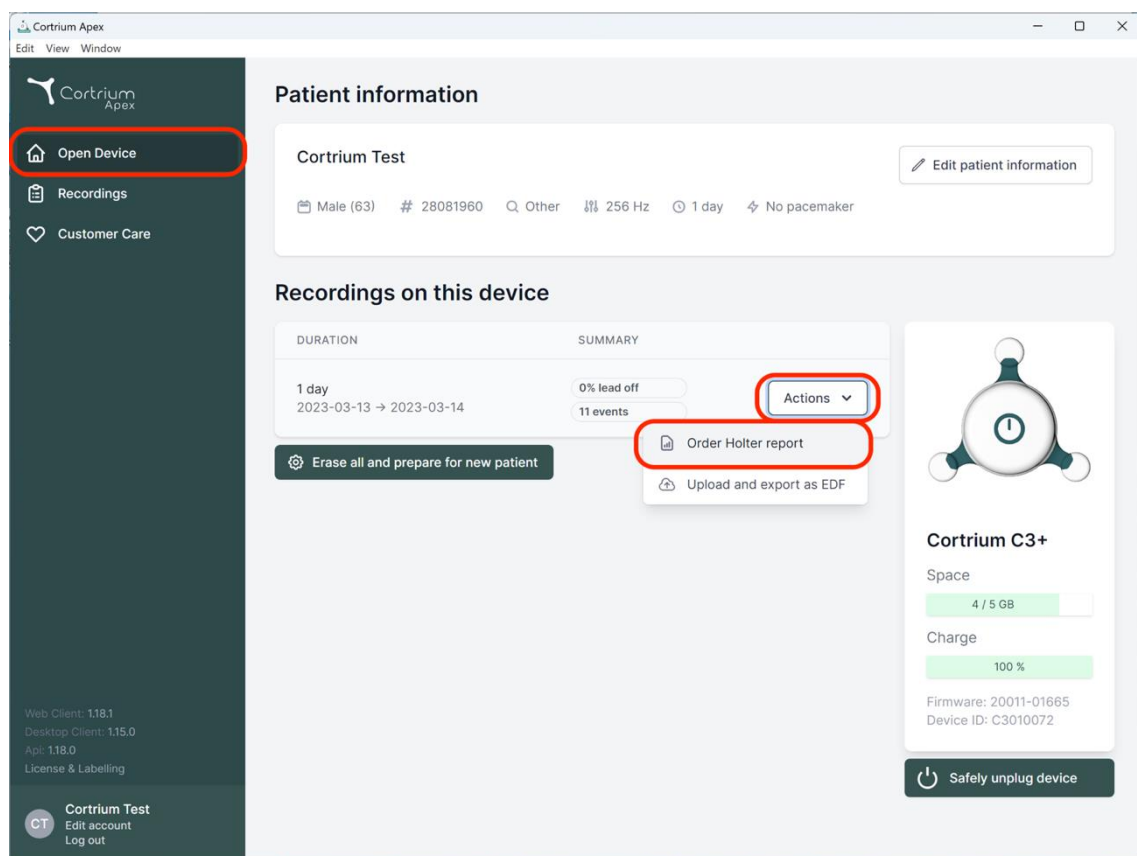
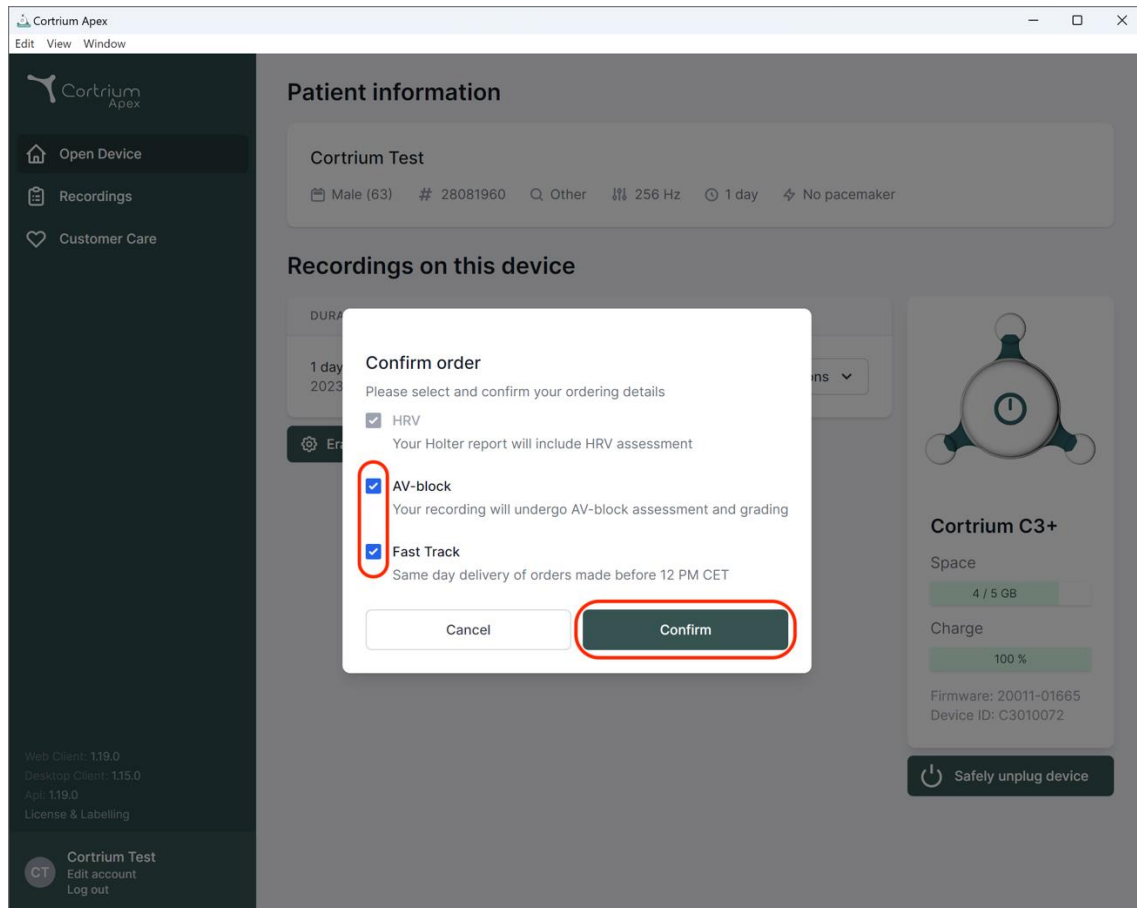
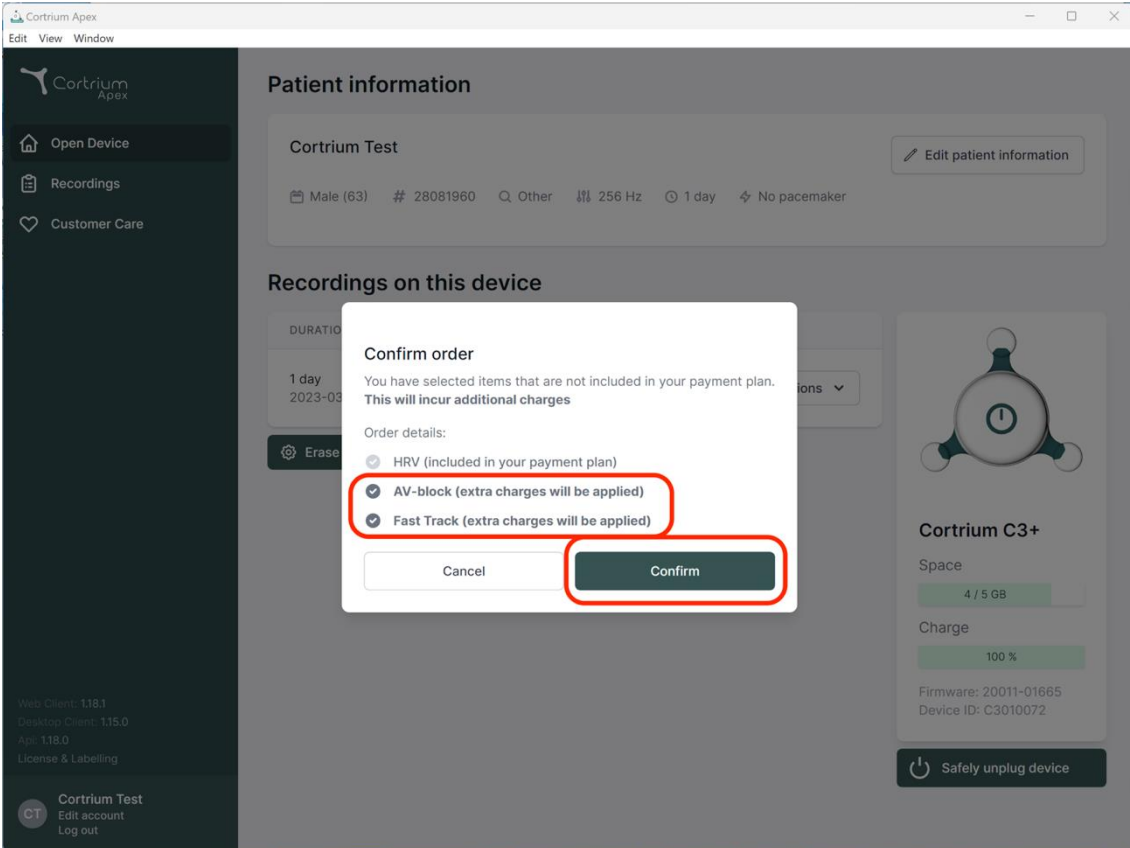
Erase

Confirm order

You have selected items that are not included in your payment plan. This will incur additional charges

Order details:

- HRV (included in your payment plan)
- AV-block (extra charges will be applied)
- Fast Track (extra charges will be applied)

Cancel Confirm

Cortrium C3+

Space

4 / 5 GB

Charge

100 %

Firmware: 20011-01665

Device ID: C3010072

Safely unplug device

Web Client: 118.1

Desktop Client: 115.0

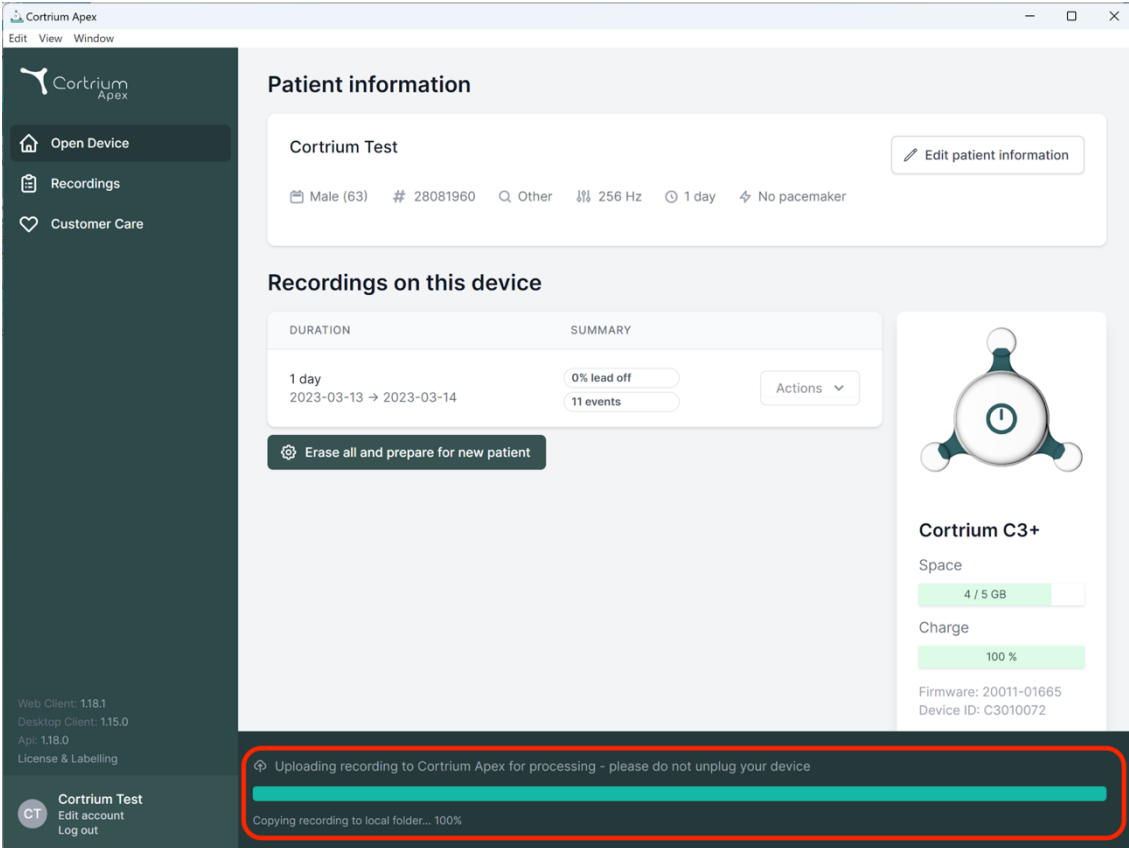
App: 118.0

License & Labelling

Cortrium Test

CT Edit account Log out

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The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, the 'Recordings on this device' section shows a recording with a 0% lead off and 11 events. A red box highlights a progress bar at the bottom of the interface, indicating 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. On the right, a device status card for 'Cortrium C3+' shows 4/5 GB space and 100% charge.

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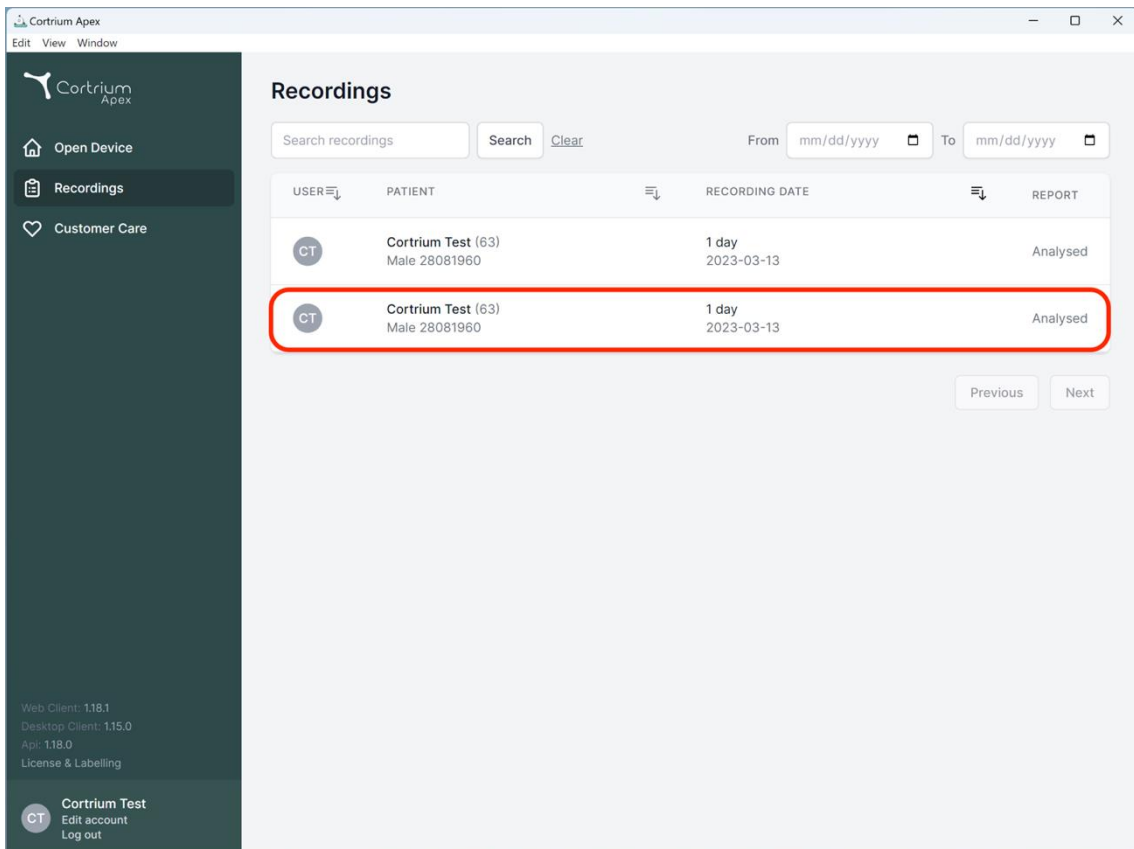
Cortrium Test
Edit account
Log out

Recordings

Search recordings Search Clear From mm/dd/yyyy To mm/dd/yyyy

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Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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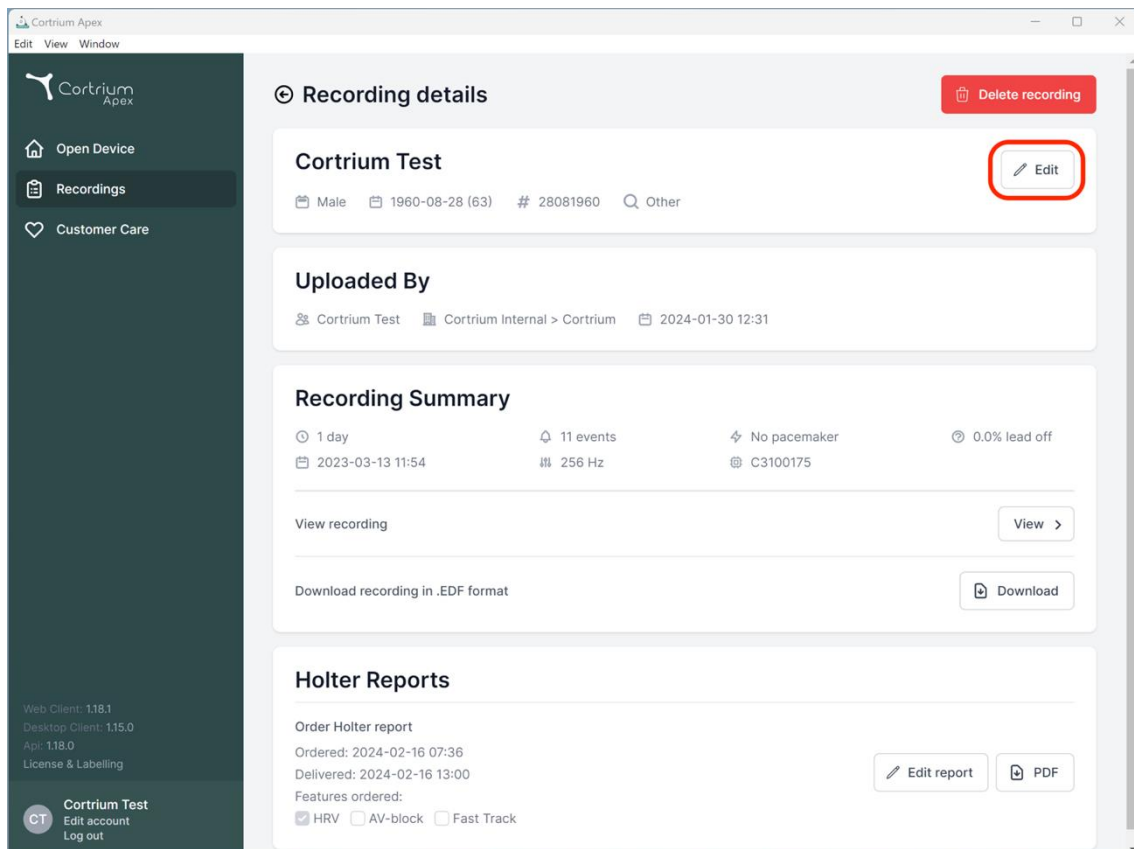
Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

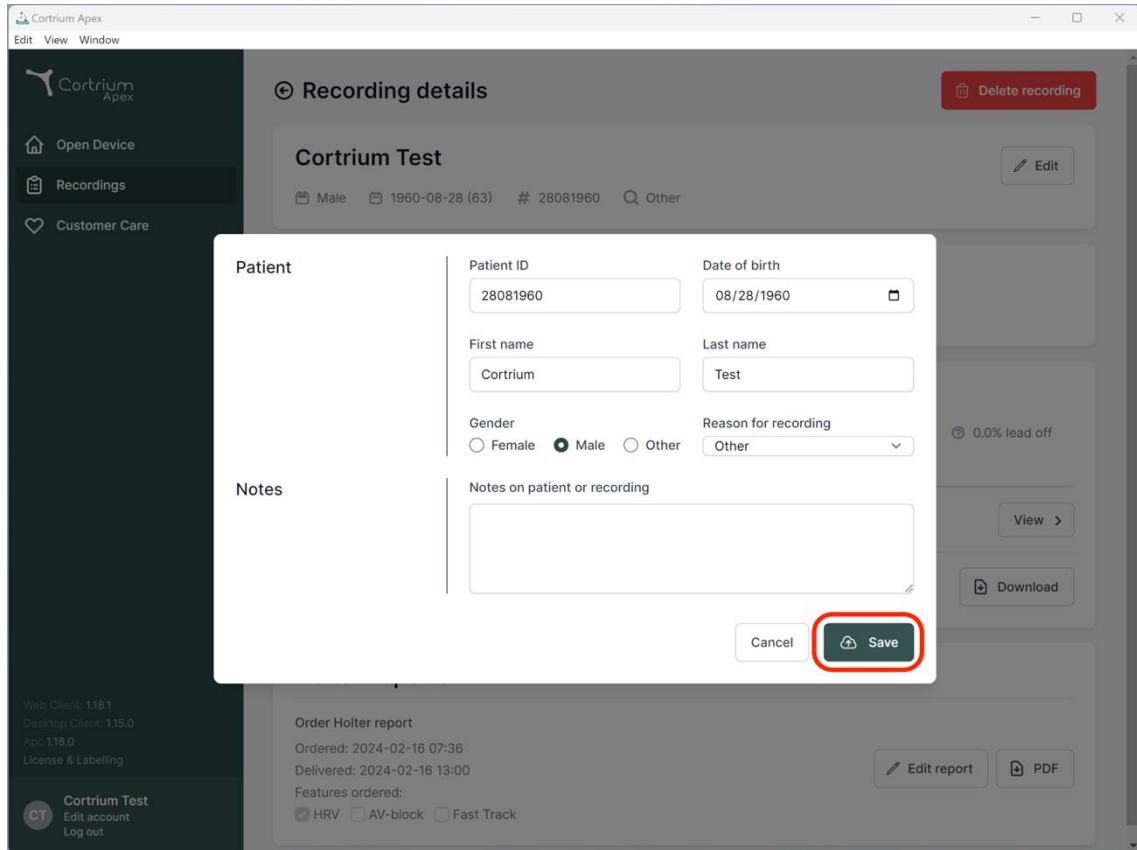
View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report
 Ordered: 2024-02-16 07:36
 Delivered: 2024-02-16 13:00
 Features ordered:
 HRV AV-block Fast Track Edit report PDF

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
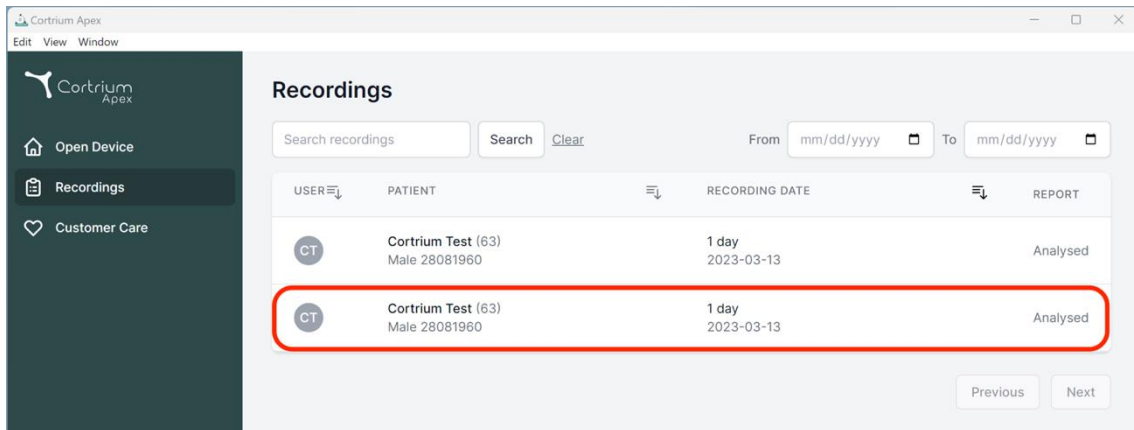
Recordings

Search recordings Search From To

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

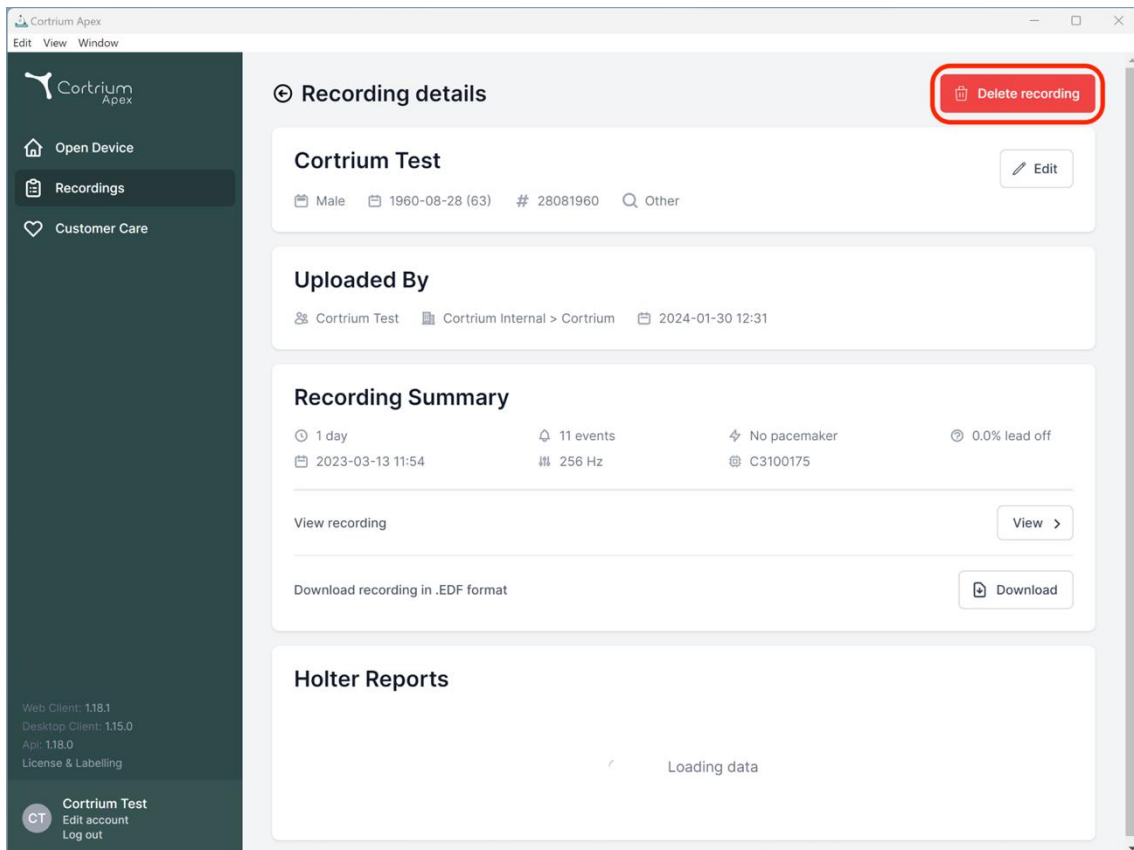Previous Next

Recording details

Cortrium Test

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
 2023-03-13 11:54 256 Hz C3100175

View recording

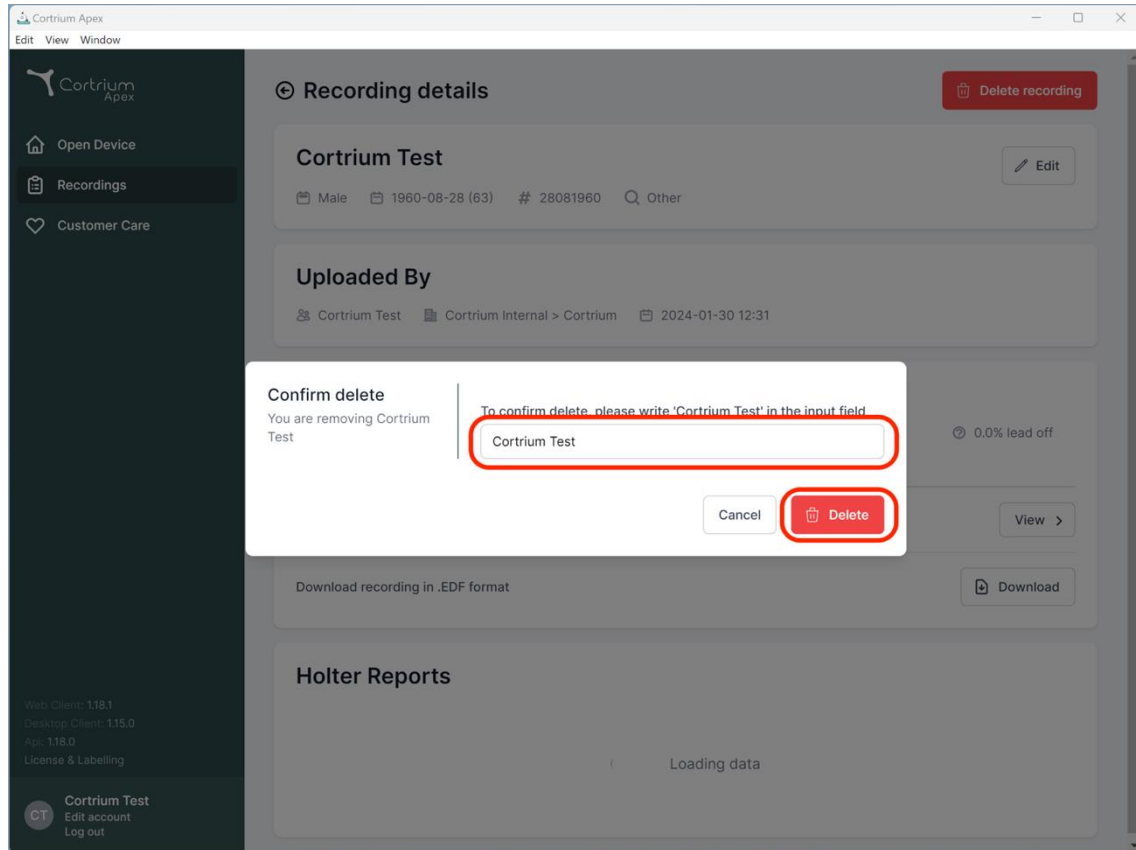
Download recording in .EDF format

Holter Reports

Loading data

Web Client: 1.18.1
 Desktop Client: 1.15.0
 App: 1.18.0
 License & Labelling

CT Cortrium Test
 Edit account
 Log out

The screenshot shows the Cortrium Apex web application interface. A 'Confirm delete' dialog box is open in the foreground, overlaid on the 'Recording details' page. The dialog box contains the following text and elements:

- Confirm delete**
- You are removing Cortrium Test
- To confirm delete, please write 'Cortrium Test' in the input field
- An input field containing the text 'Cortrium Test' (highlighted with a red circle).
- Buttons for 'Cancel' and 'Delete' (the 'Delete' button is highlighted with a red circle).

The background page, 'Recording details', shows the following information:

- Recording details** (with a 'Delete recording' button)
- Cortrium Test** (with an 'Edit' button)
- Male, 1960-08-28 (63), # 28081960, Other
- Uploaded By**
- Cortrium Test, Cortrium Internal > Cortrium, 2024-01-30 12:31
- 0.0% lead off
- Download recording in .EDF format (with a 'Download' button)
- Holter Reports**
- Loading data

The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The bottom left corner shows user information for 'Cortrium Test' (CT) with options to 'Edit account' and 'Log out'. The top left of the browser window shows 'Cortrium Apex' and menu options 'Edit', 'View', and 'Window'.

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
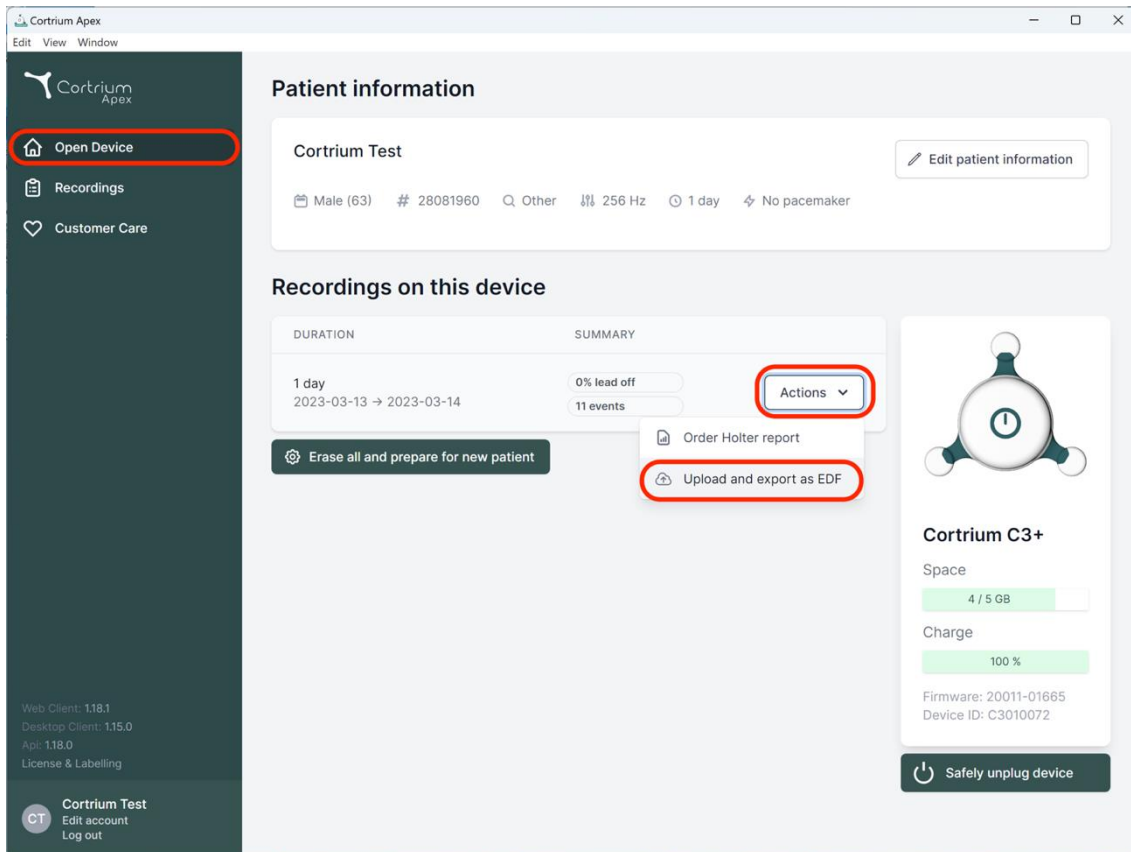
The screenshot displays the Cortrium Apex web application interface. On the left sidebar, the 'Open Device' button is highlighted with a red box. The main content area shows 'Patient information' for 'Cortrium Test', including details like 'Male (63)', '# 28081960', 'Other', '256 Hz', '1 day', and 'No pacemaker'. Below this, the 'Recordings on this device' section shows a recording for '1 day' (2023-03-13 to 2023-03-14) with '0% lead off' and '11 events'. An 'Actions' dropdown menu is open, with the 'Upload and export as EDF' option highlighted by a red box. On the right, the 'Cortrium C3+' device status is shown, including 'Space: 4 / 5 GB', 'Charge: 100%', and 'Firmware: 20011-01665'. A 'Safely unplug device' button is visible at the bottom right.

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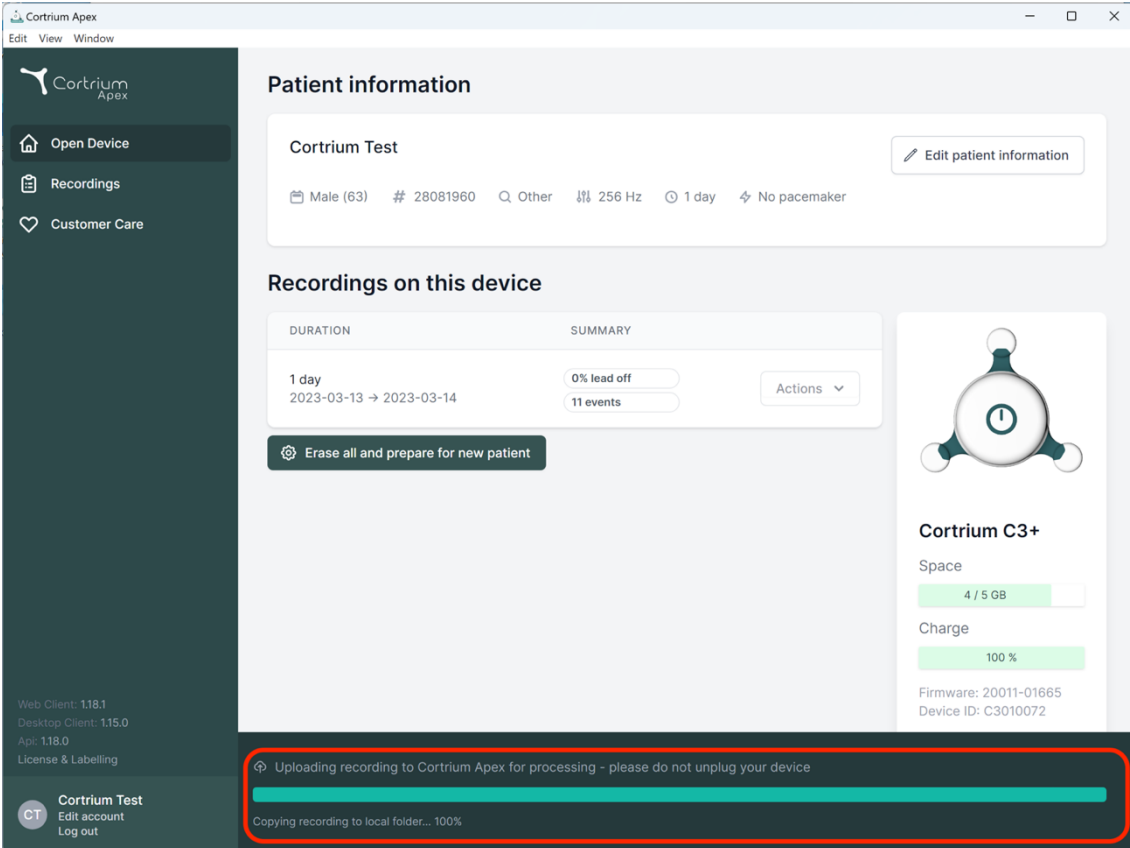Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device


DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB
Charge: 100 %
Firmware: 20011-01665
Device ID: C3010072

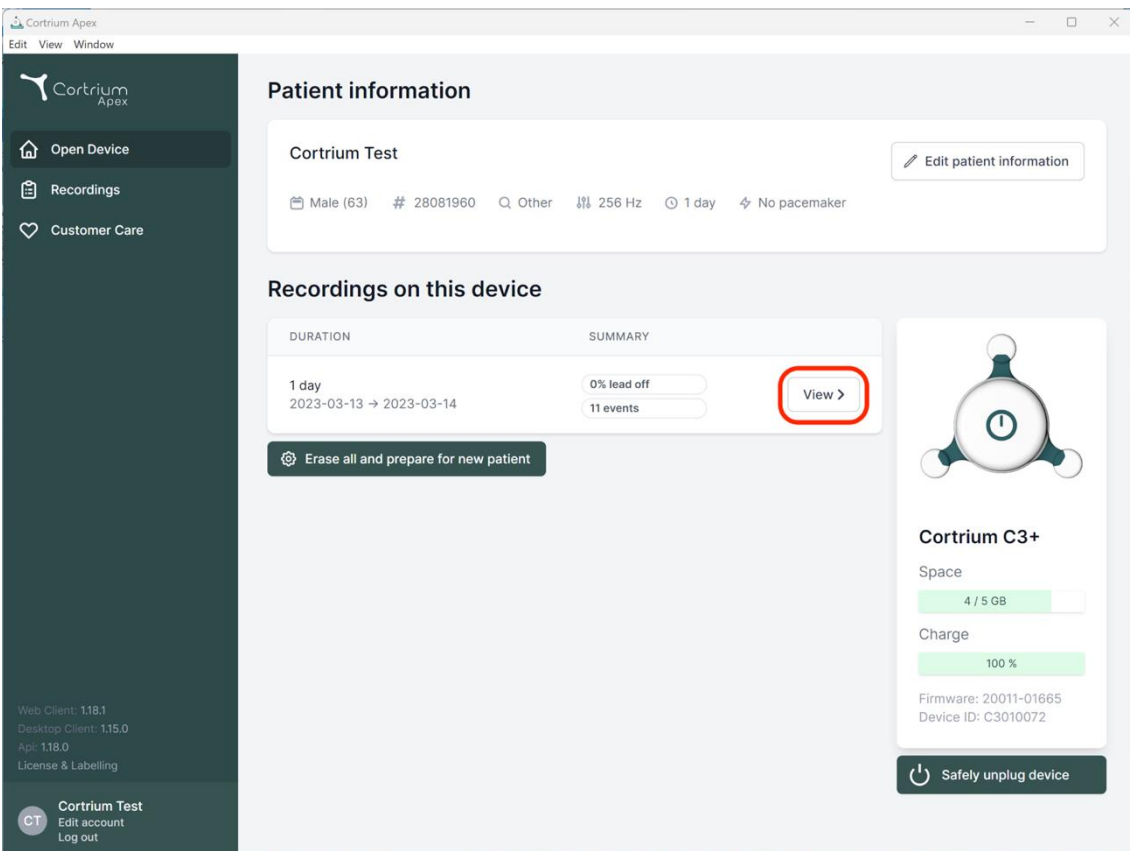Uploading recording to Cortrium Apex for processing - please do not unplug your device
Copying recording to local folder... 100%

Cortrium Apex

Edit View Window

Open Device
Recordings
Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
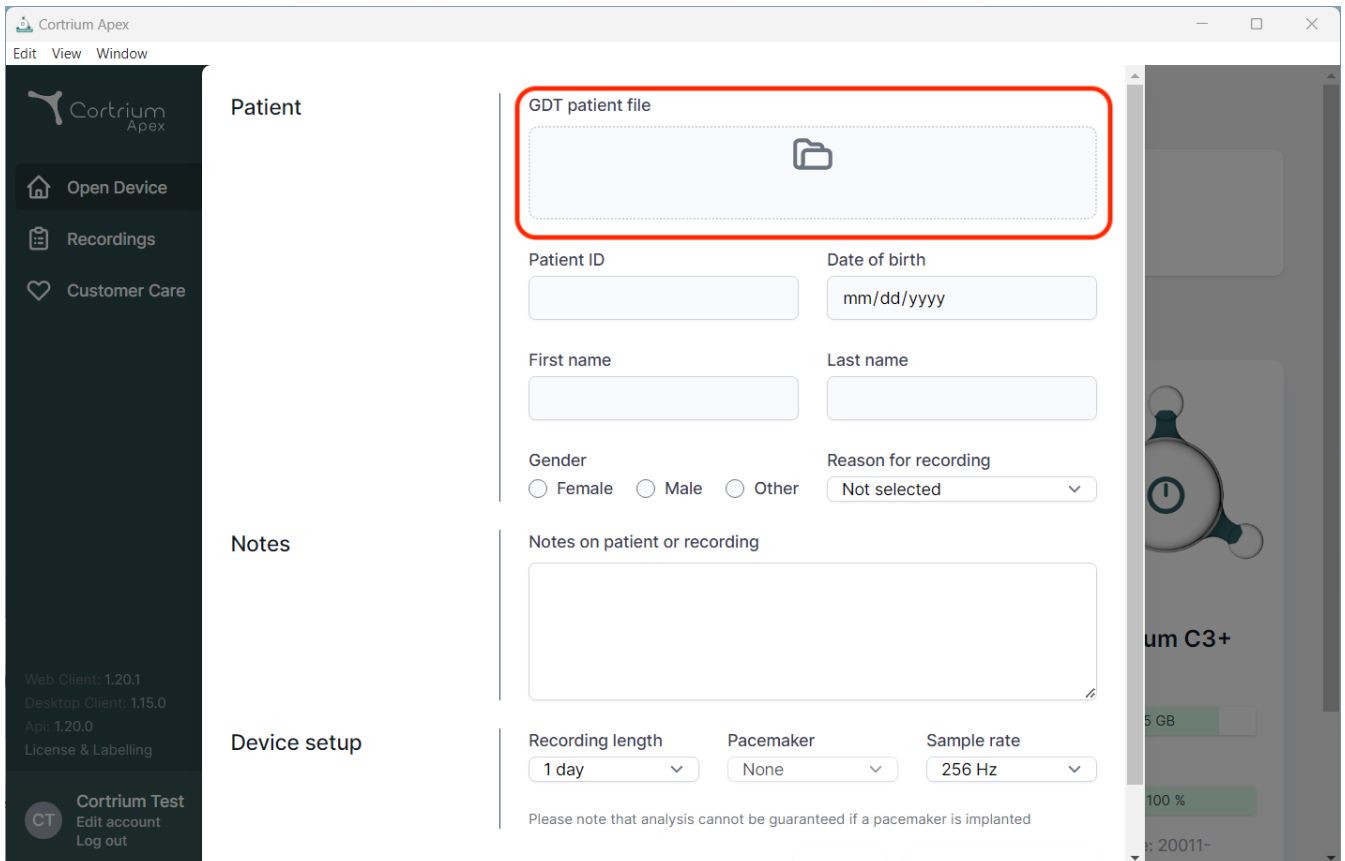
The screenshot displays the Cortrium Apex web application interface. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a form for entering patient information. The 'GDT patient file' field is highlighted with a red box and contains a folder icon. Below this are fields for Patient ID, Date of birth (mm/dd/yyyy), First name, Last name, Gender (Female, Male, Other), and Reason for recording (Not selected). The Notes section has a text area for 'Notes on patient or recording'. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. The left sidebar contains navigation options: Open Device, Recordings, Customer Care, and user information for 'Cortrium Test'.

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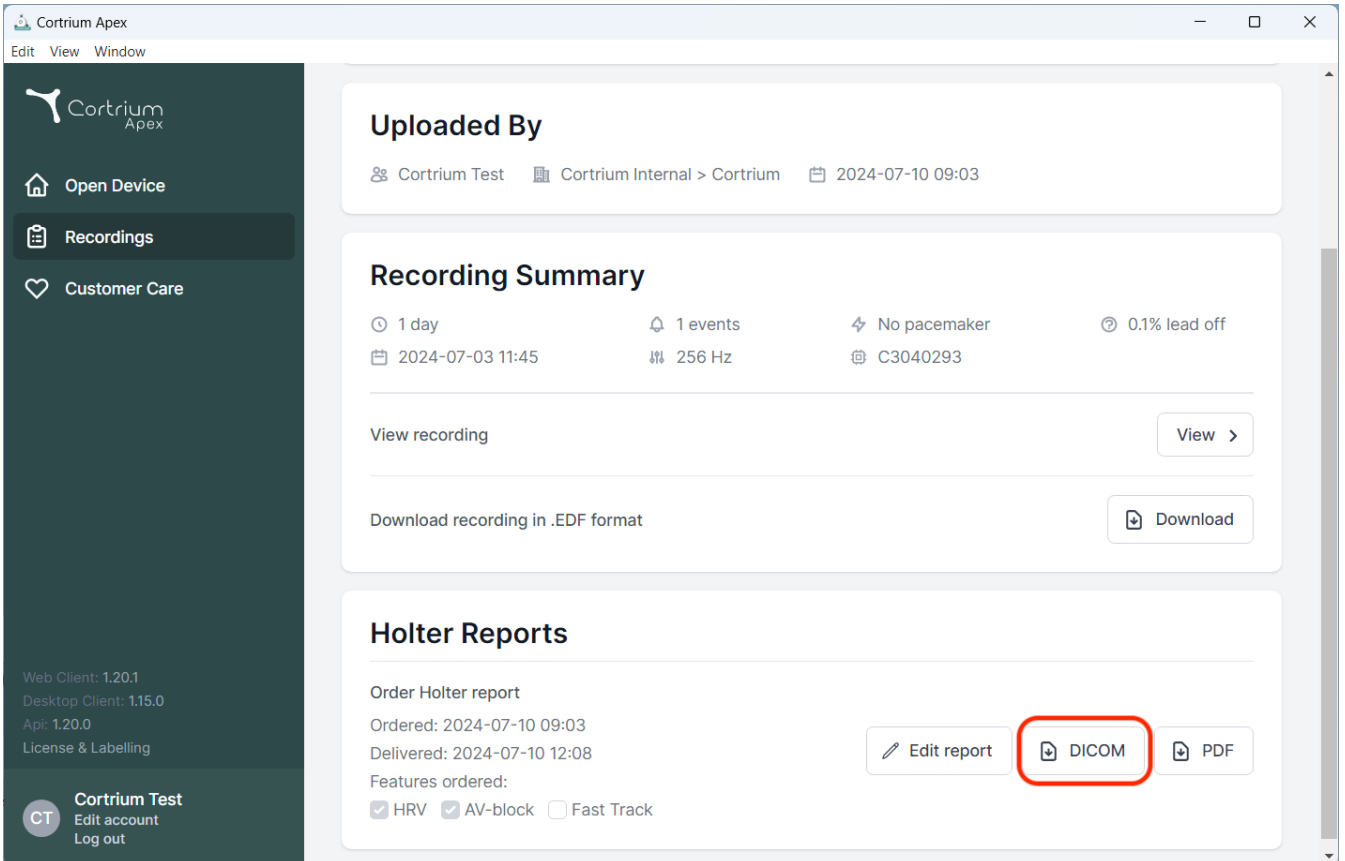
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' (2024-07-03 11:45), '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off' (C3040293). It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists ordered features: 'HRV', 'AV-block', and 'Fast Track'. Three buttons are visible: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

Footer information includes: Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0, License & Labelling, and user 'CT Cortrium Test' with 'Edit account' and 'Log out' options.

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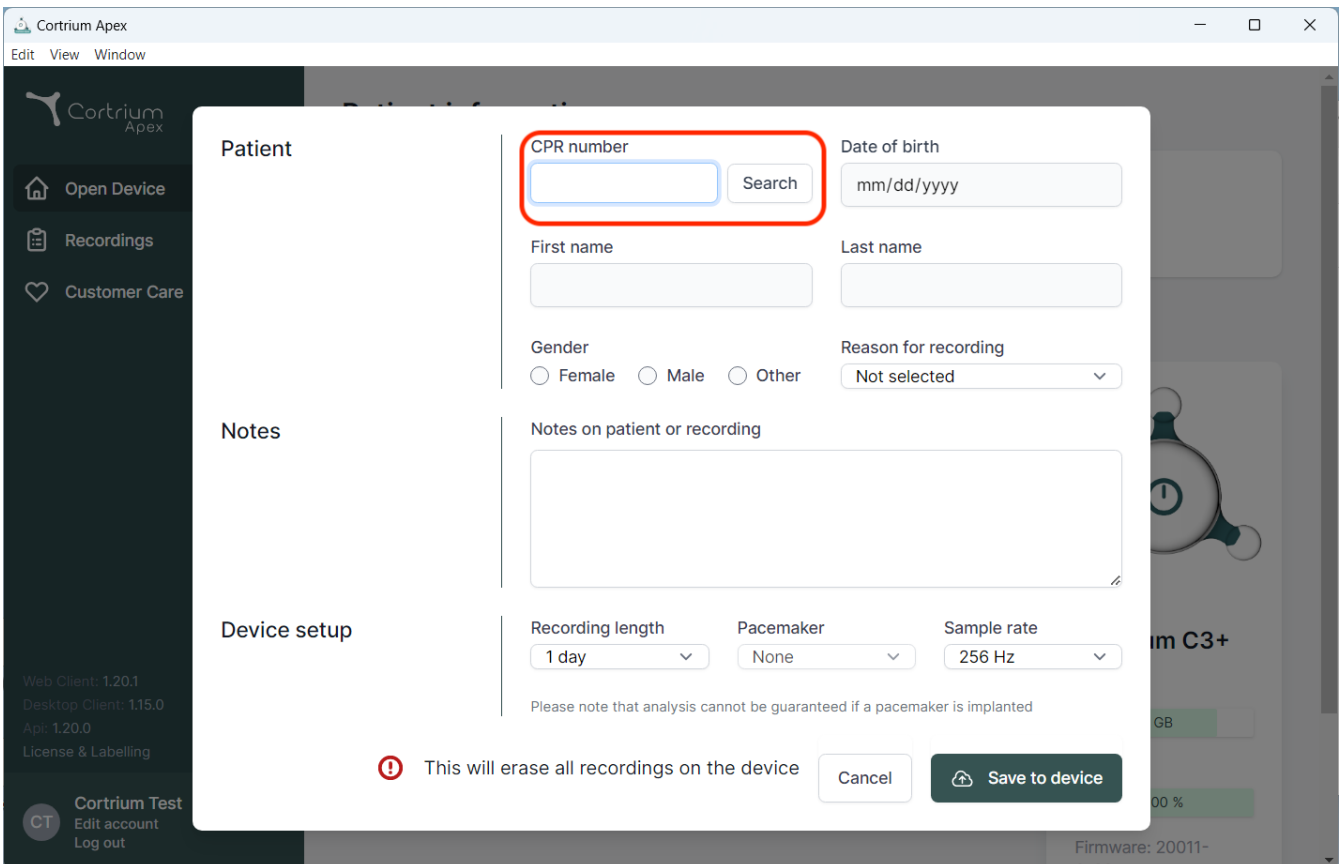The screenshot displays the Cortrium Apex software interface. A modal window titled "Patient" is open, showing a form for entering patient information. The "CPR number" field is highlighted with a red box. The form includes the following fields and options:

- CPR number:** A text input field with a "Search" button next to it.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently set to "Not selected".
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the form, a warning message with a red exclamation mark icon states: "This will erase all recordings on the device". At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

09/2024

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dānija
E-pasts: info@cortrium.com • Tīmekļa vietne: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123