# Cortrium Apex 1.20

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
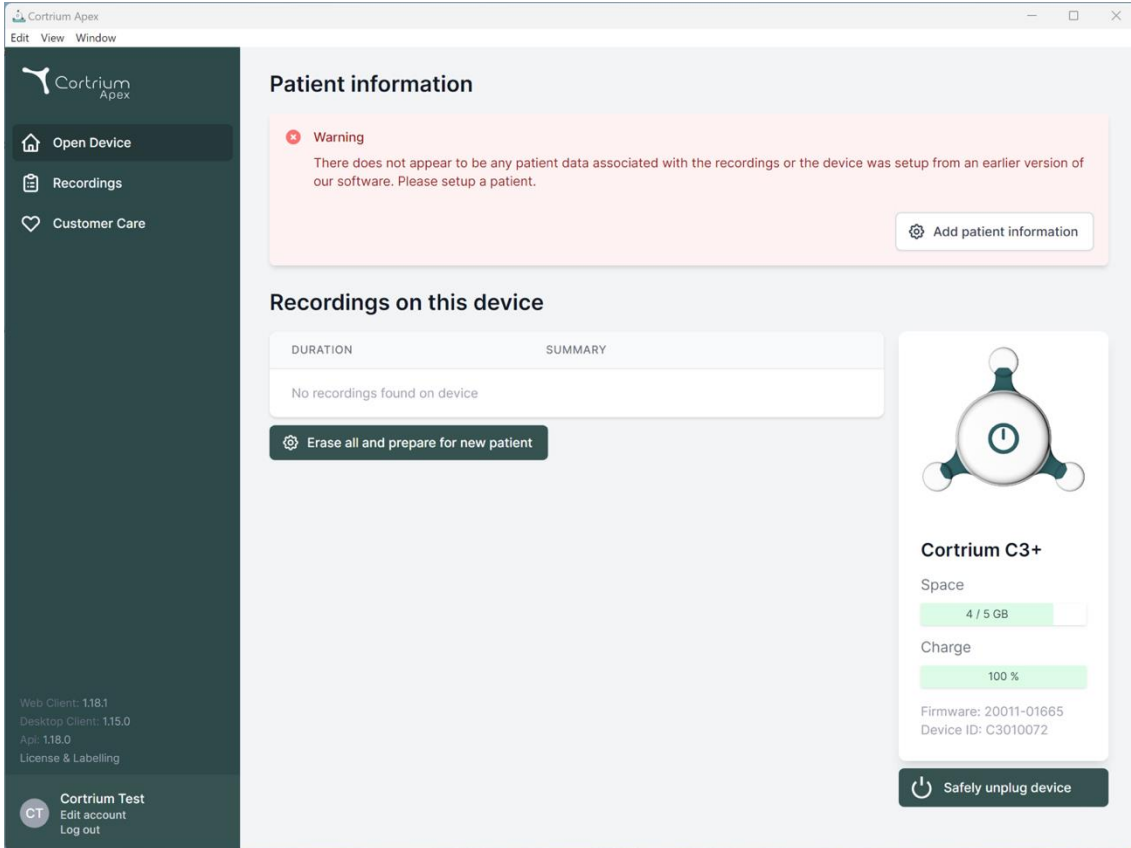
The screenshot shows the Cortrium Apex web interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, it shows version information (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, License & Labelling) and a 'Cortrium Test' section with 'Edit account' and 'Log out' links.

The main content area is titled 'Patient information' and features a red warning box: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' Below this is a button labeled 'Add patient information'.

Below the warning is a section titled 'Recordings on this device' with a table with columns 'DURATION' and 'SUMMARY'. The table is empty, showing 'No recordings found on device'. Below the table is a button labeled 'Erase all and prepare for new patient'.

On the right side of the main area, there is a device status card for 'Cortrium C3+'. It shows a battery icon, 'Space: 4 / 5 GB', 'Charge: 100%', 'Firmware: 20011-01665', and 'Device ID: C3010072'. At the bottom of the card is a button labeled 'Safely unplug device'.

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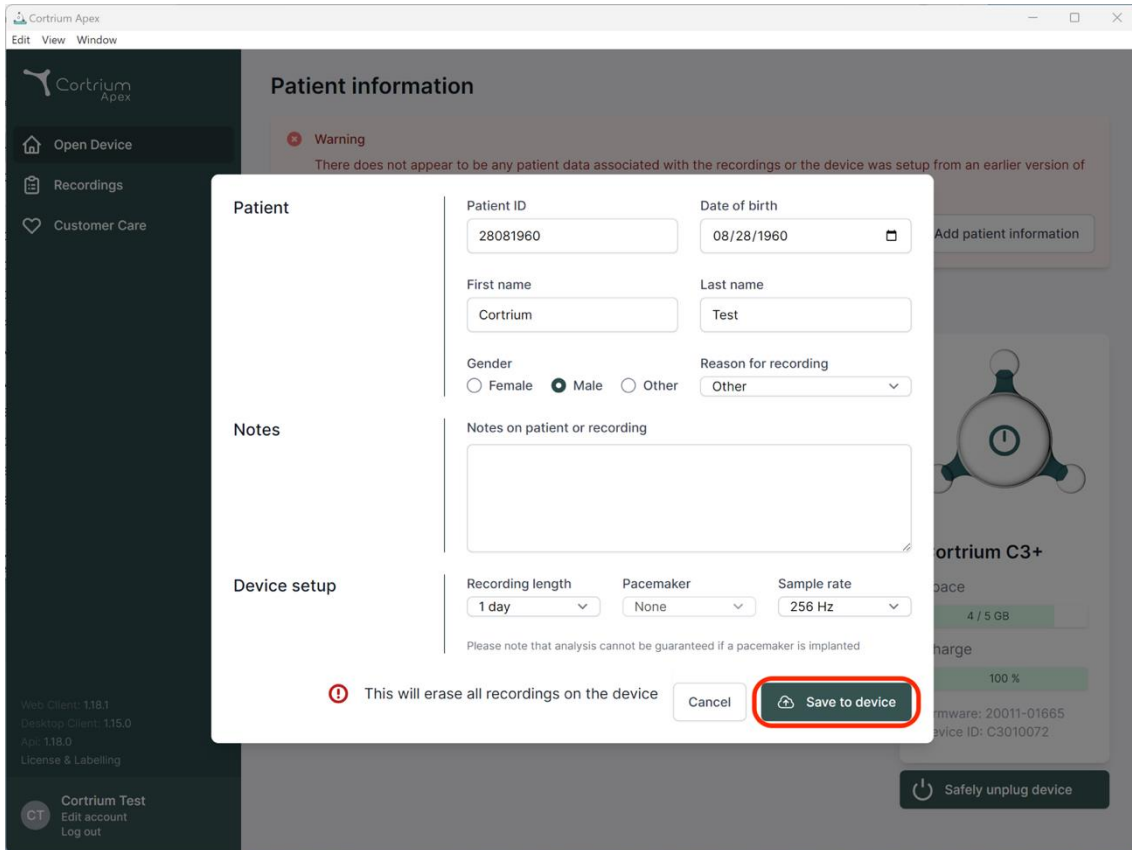Edit View Window

### Patient information

**Warning**  
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of...

**Patient**

Patient ID: 28081960  
Date of birth: 08/28/1960

First name: Cortrium  
Last name: Test

Gender:  Female  Male  Other  
Reason for recording: Other

**Notes**

Notes on patient or recording

**Device setup**

Recording length: 1 day  
Pacemaker: None  
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

**⚠ This will erase all recordings on the device**

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
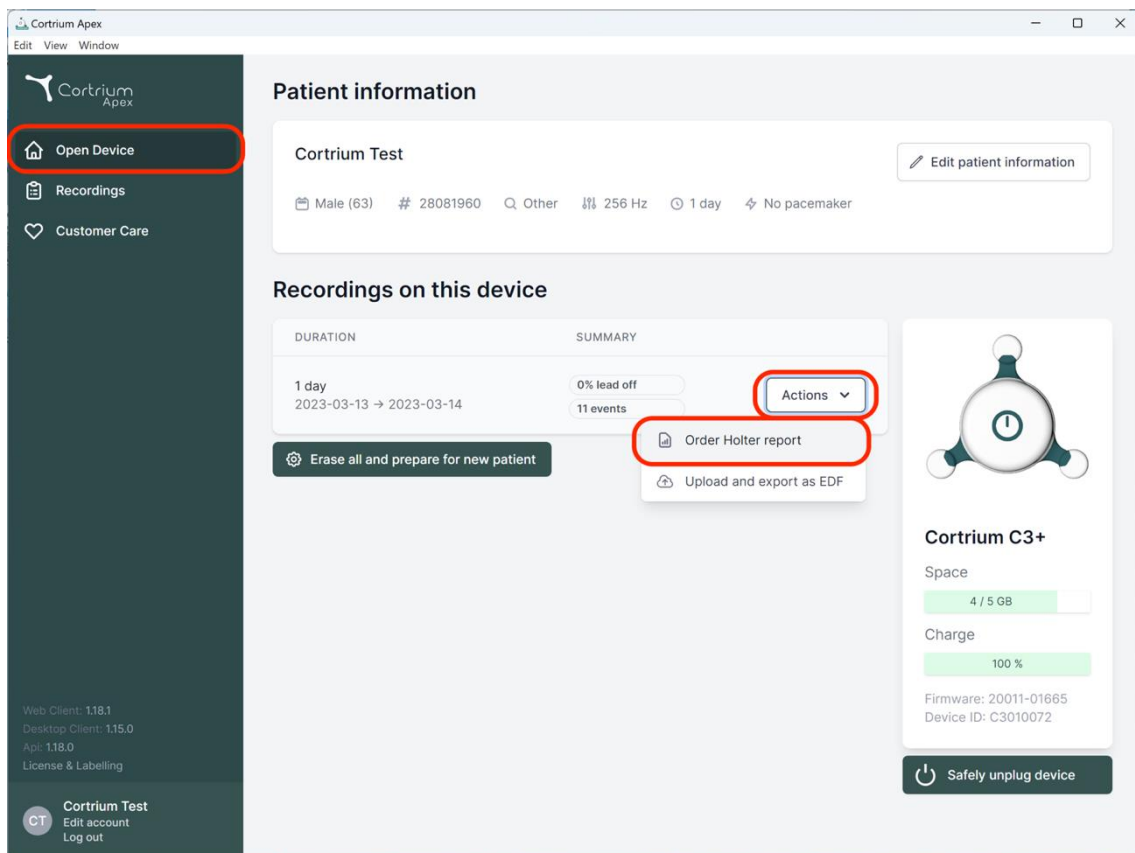
The screenshot displays the Cortrium Apex web application interface. On the left, a dark sidebar contains navigation options: 'Open Device' (highlighted with a red box), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including patient demographics and device settings. Below this, the 'Recordings on this device' section shows a recording for '1 day' from 2023-03-13 to 2023-03-14, with a summary of '0% lead off' and '11 events'. An 'Actions' dropdown menu (highlighted with a red box) is open, showing 'Order Holter report' (highlighted with a red box) and 'Upload and export as EDF'. On the right, a device status card for 'Cortrium C3+' shows 'Space' at 4/5 GB and 'Charge' at 100%. A 'Safely unplug device' button is at the bottom right. The footer includes version information and a user profile for 'Cortrium Test'.

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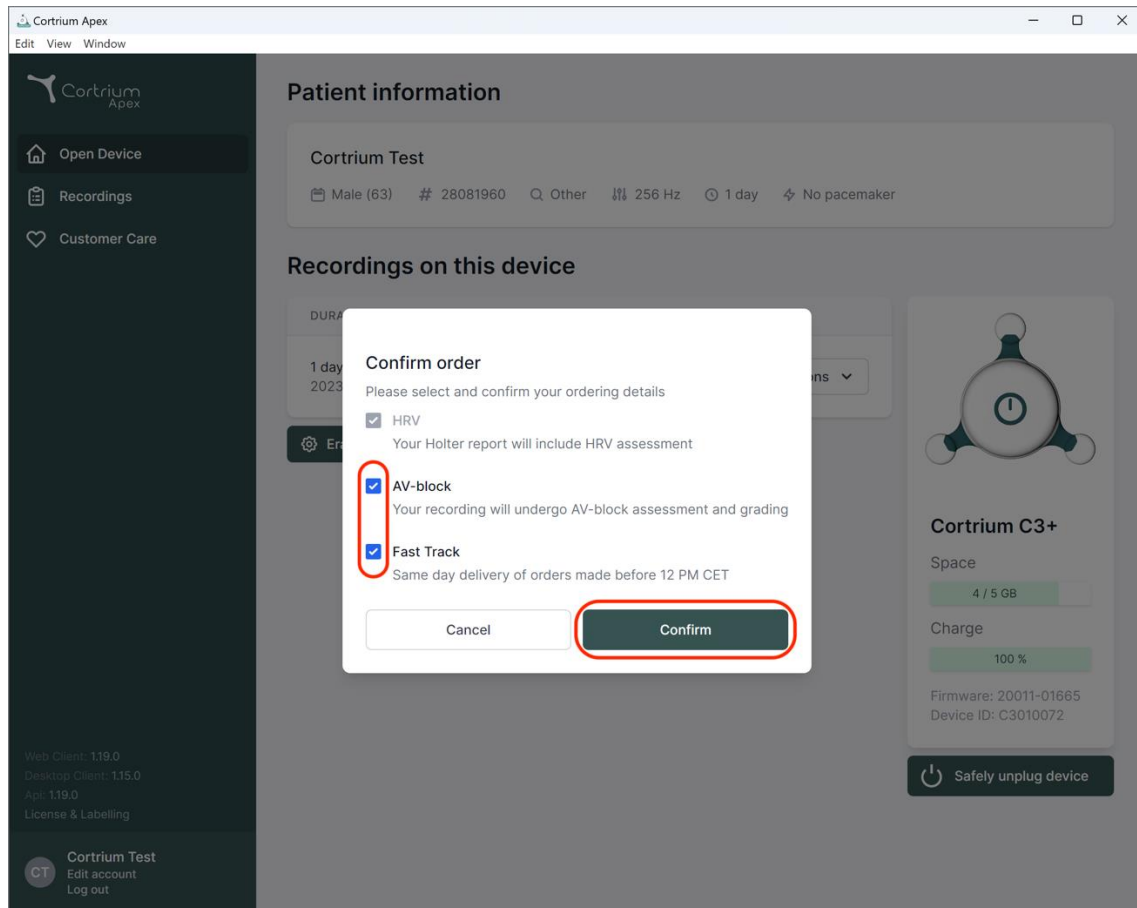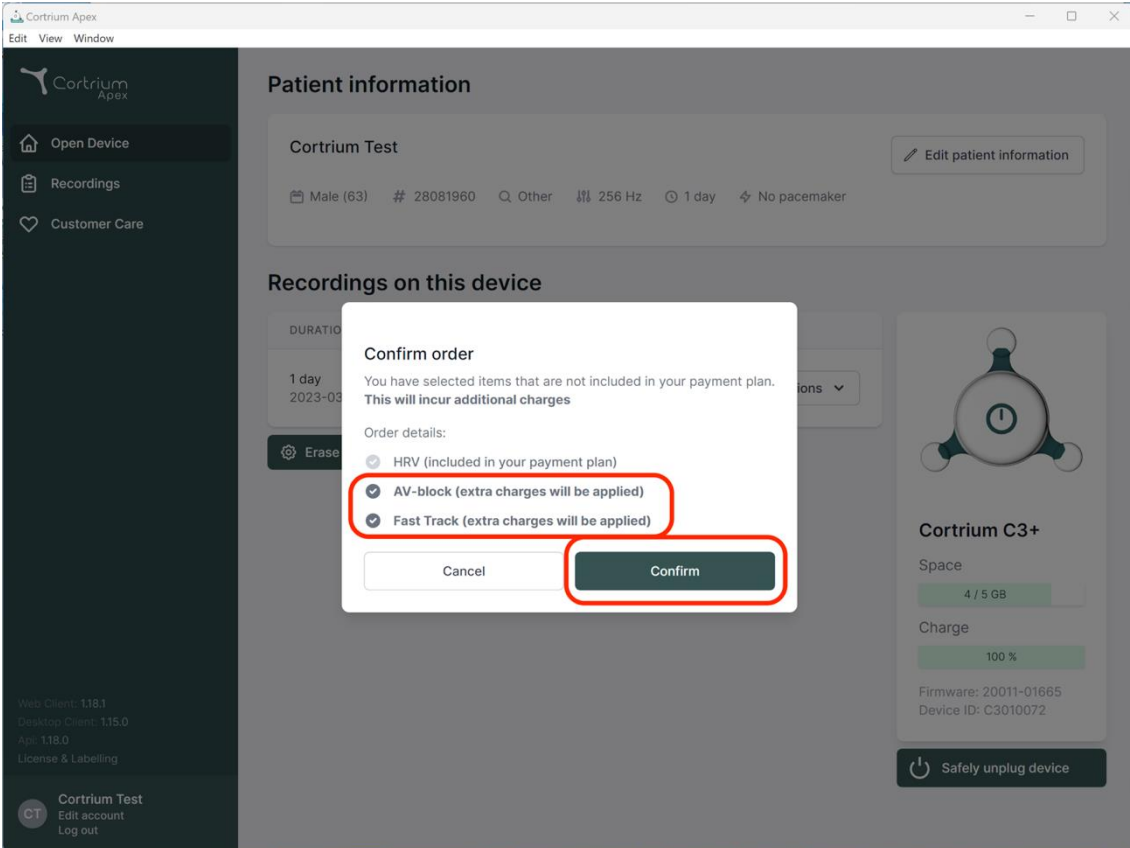
Erase

**Confirm order**

You have selected items that are not included in your payment plan.  
**This will incur additional charges**

Order details:

- HRV (included in your payment plan)
- AV-block (extra charges will be applied)
- Fast Track (extra charges will be applied)

Cancel Confirm

Cortrium C3+

Space

4 / 5 GB

Charge

100 %

Firmware: 20011-01665


Device ID: C3010072

Safely unplug device

Web Client: 118.1  
Desktop Client: 115.0  
App: 118.0  
License & Labelling

Cortrium Test  
CT Edit account  
Log out

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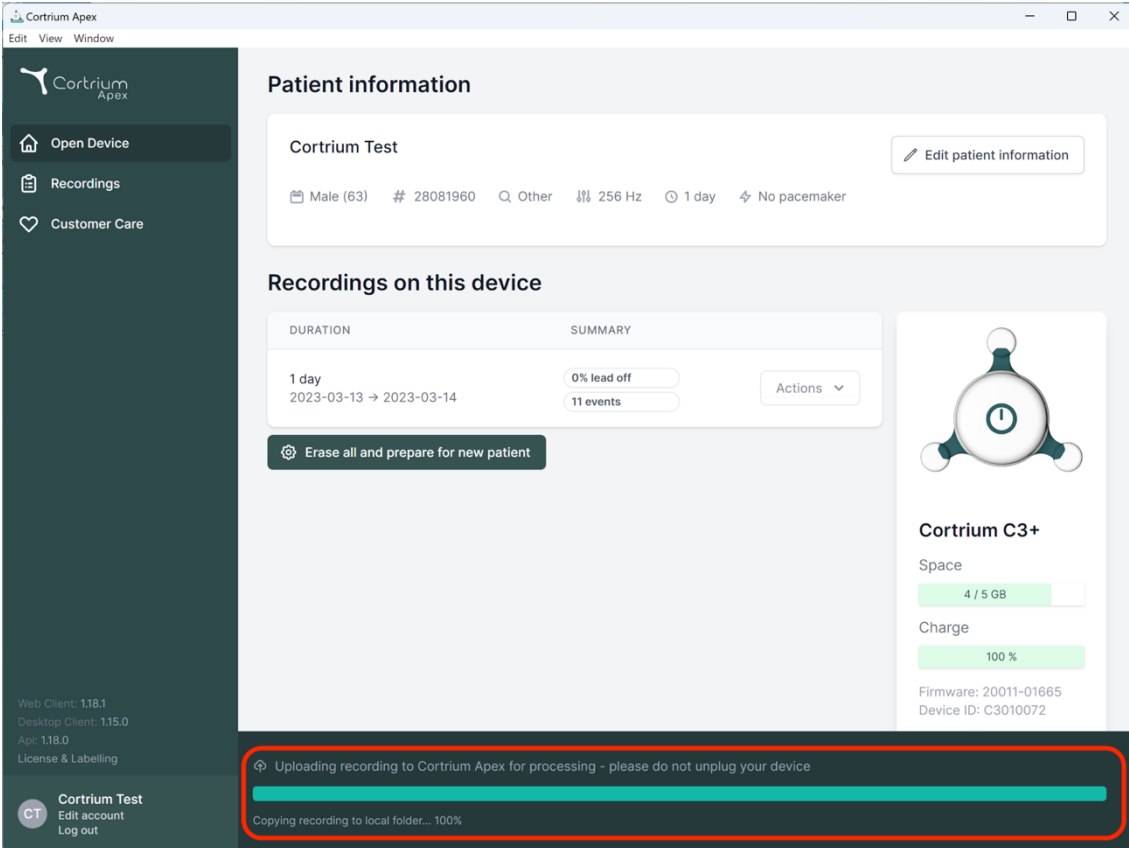The screenshot displays the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window'. The left sidebar contains 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, the 'Recordings on this device' section shows a recording with a 0% lead off and 11 events. A red box highlights a progress bar at the bottom of the interface, indicating the upload status: 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. The right sidebar shows device status for 'Cortrium C3+', including space (4 / 5 GB) and charge (100%).

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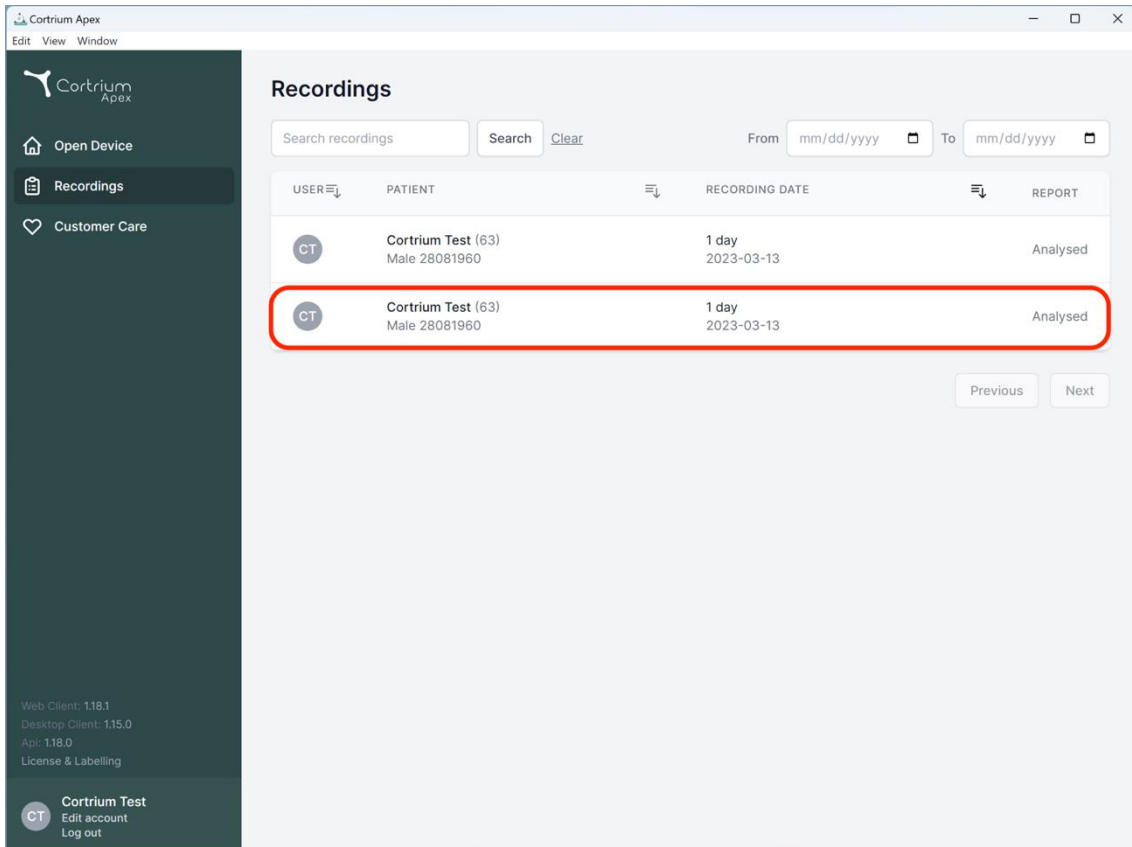
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

Cortrium Apex  
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1  
Desktop Client: 1.15.0  
App: 1.18.0  
License & Labelling

**CT** Cortrium Test  
Edit account  
Log out

### Recording details

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

#### Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

#### Recording Summary

1 day 11 events No pacemaker 0.0% lead off  
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

#### Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36  
Delivered: 2024-02-16 13:00

Features ordered:  
 HRV  AV-block  Fast Track

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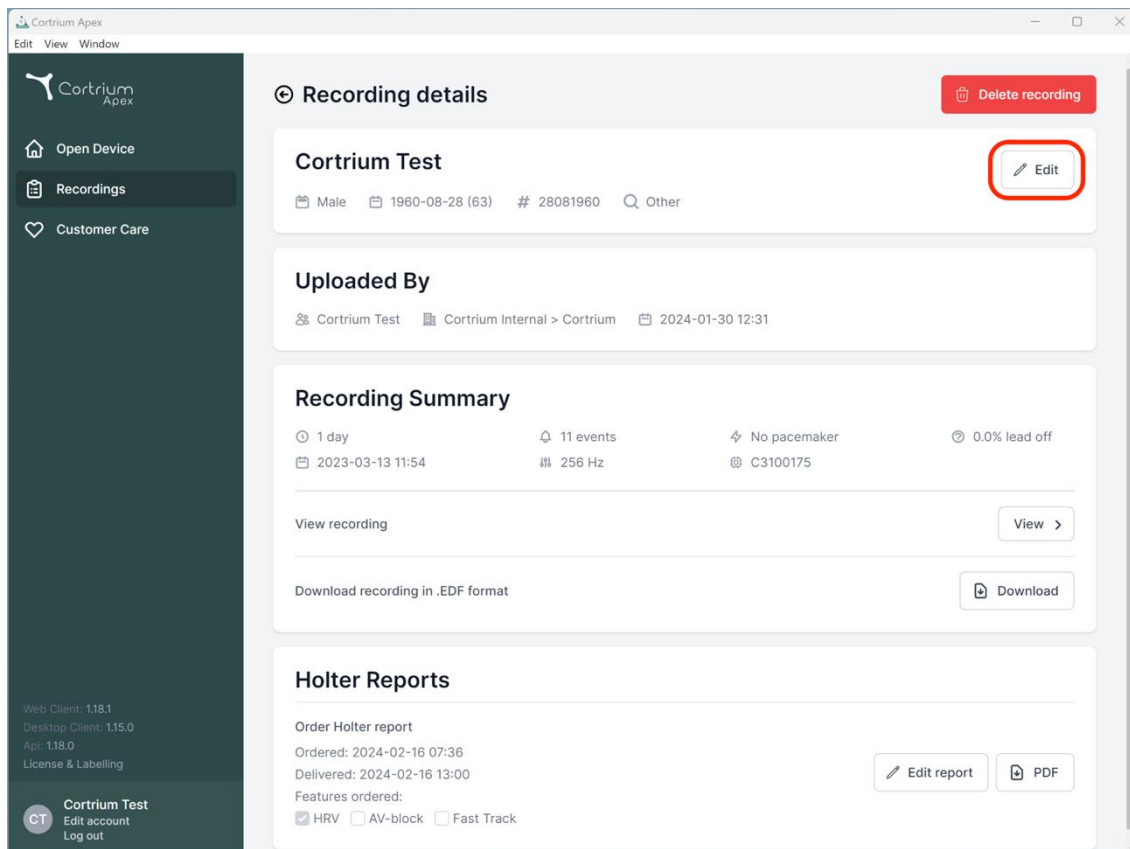
**Recording details** Delete recording

**Cortrium Test** Edit

Male 1960-08-28 (63) # 28081960 Other

**Uploaded By**

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

**Recording Summary**

1 day 11 events No pacemaker 0.0% lead off  
 2023-03-13 11:54 256 Hz C3100175

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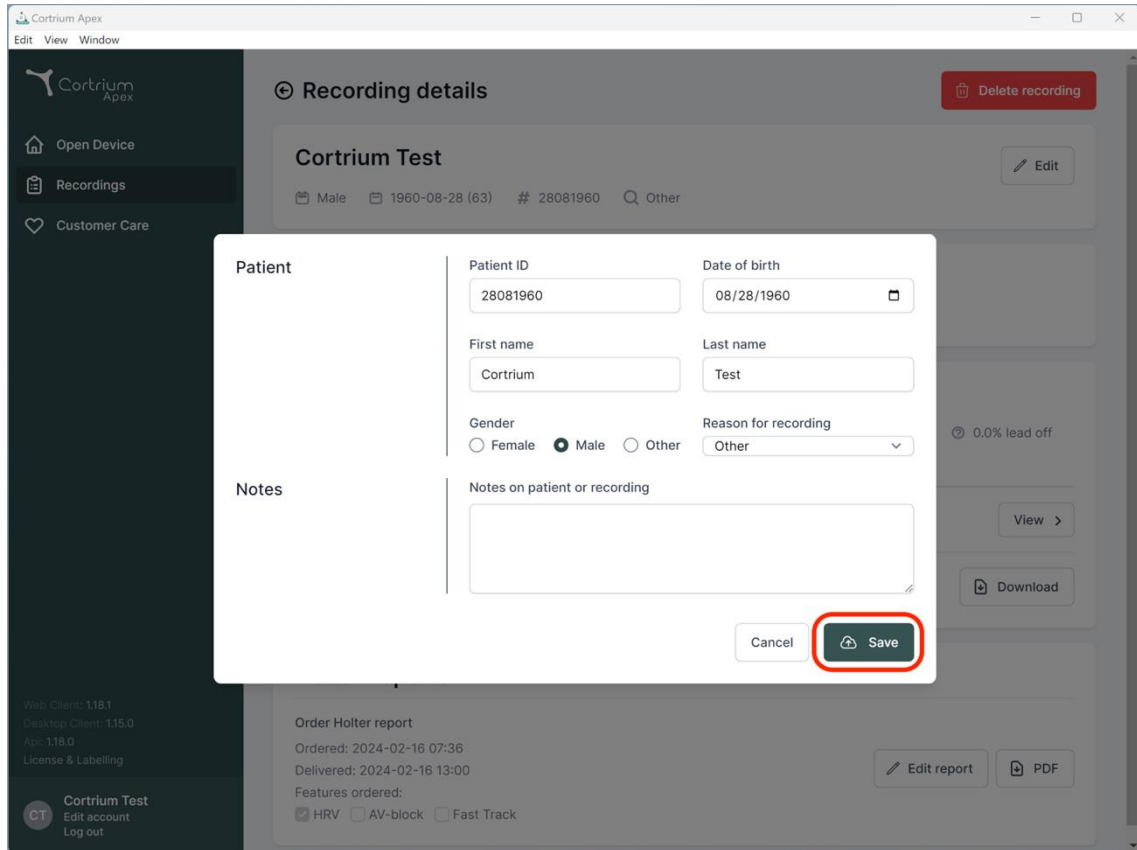
Ordered: 2024-02-16 07:36  
 Delivered: 2024-02-16 13:00

Features ordered:

HRV  AV-block  Fast Track Edit report PDF

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
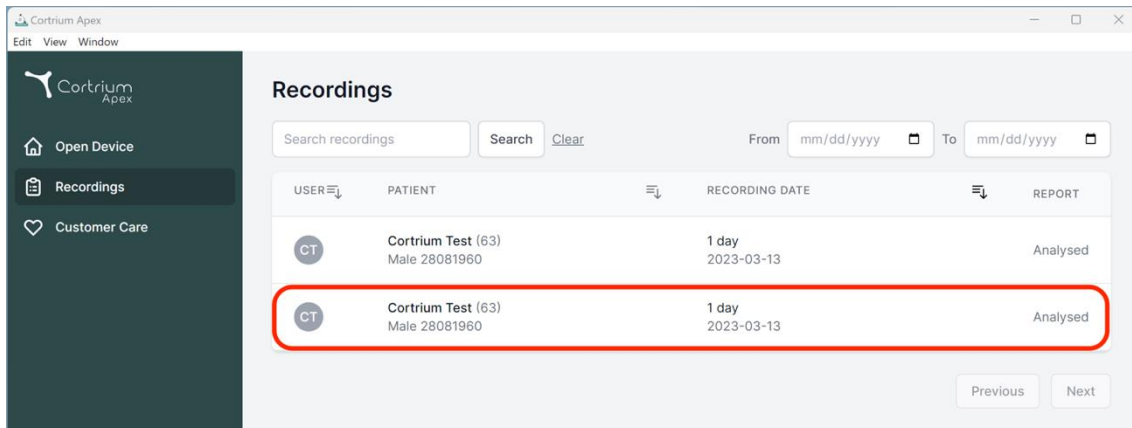
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. It features a search bar and a table with columns for USER, PATIENT, RECORDING DATE, and REPORT. Two entries are listed, both for 'Cortrium Test (63)'. The second entry is highlighted with a red box.

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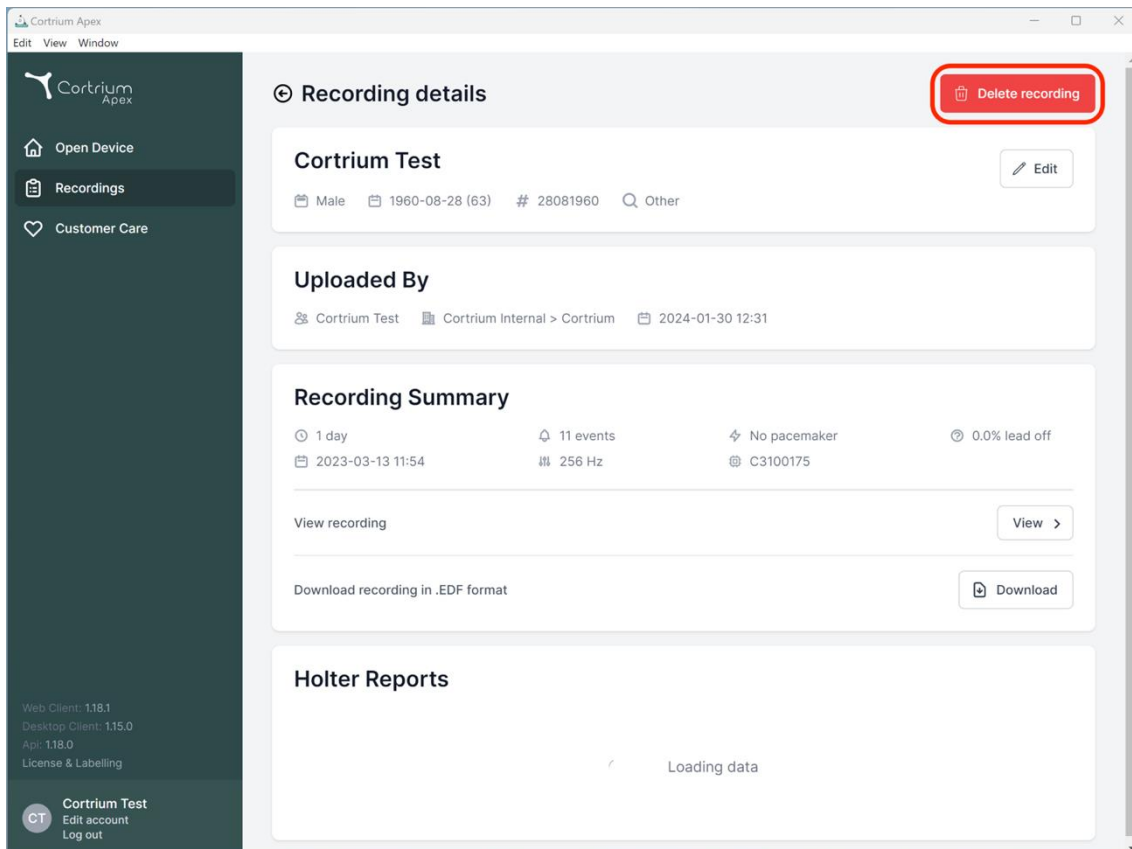The screenshot shows the 'Recording details' page for 'Cortrium Test'. It includes a 'Delete recording' button highlighted with a red box, an 'Edit' button, and sections for 'Uploaded By', 'Recording Summary', and 'Holter Reports'. The recording summary shows a 1-day recording with 11 events, no pacemaker, and a 0.0% lead off.

**Recording details**

**Cortrium Test** [Edit]

Male 1960-08-28 (63) # 28081960 [Other]

**Uploaded By**

Cortrium Test | Cortrium Internal > Cortrium | 2024-01-30 12:31

**Recording Summary**

1 day | 11 events | No pacemaker | 0.0% lead off

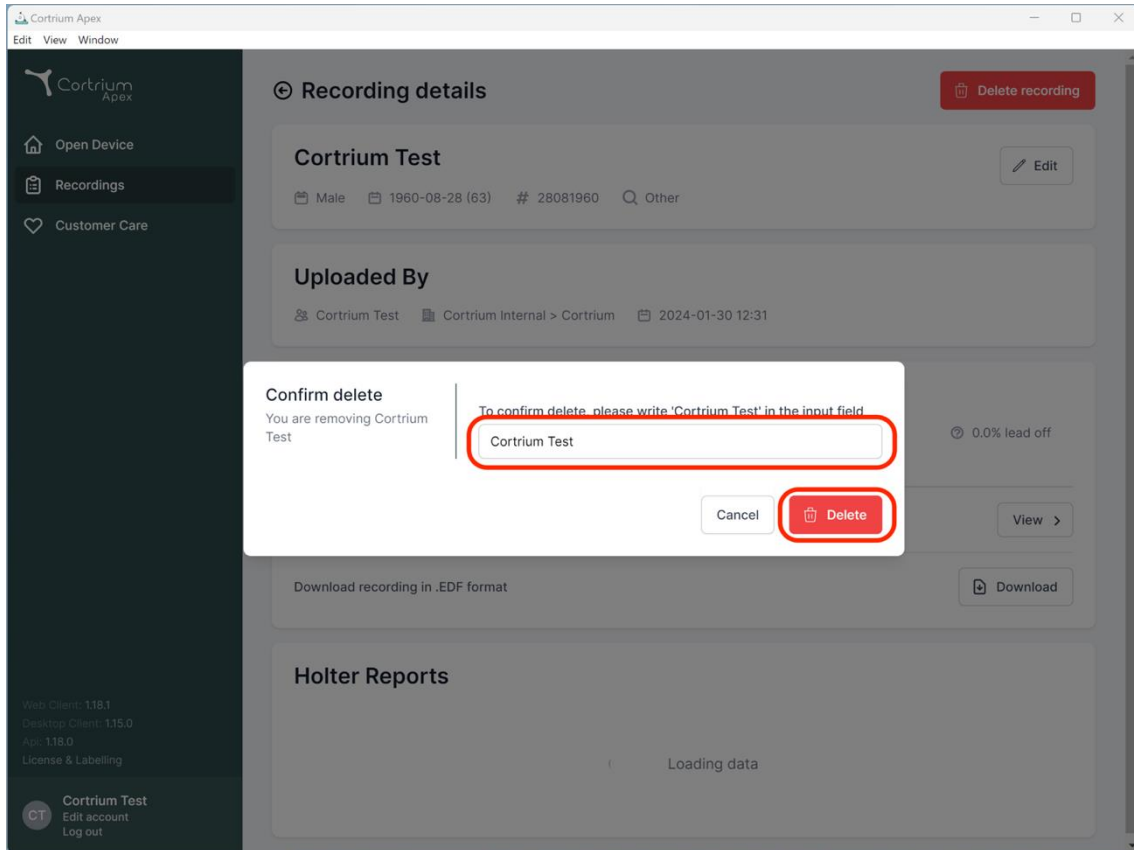
2023-03-13 11:54 | 256 Hz | C3100175

View recording [View >]

Download recording in .EDF format [Download]

**Holter Reports**

Loading data

The screenshot shows the Cortrium Apex web application interface. A 'Confirm delete' dialog box is open in the foreground, overlaid on the 'Recording details' page. The dialog box contains the following text and elements:

- Confirm delete**
- You are removing Cortrium Test
- To confirm delete, please write 'Cortrium Test' in the input field
- An input field containing the text 'Cortrium Test' (highlighted with a red circle).
- Buttons for 'Cancel' and 'Delete' (the 'Delete' button is highlighted with a red circle).

The background page, 'Recording details', shows the following information:

- Recording details** (with a 'Delete recording' button)
- Cortrium Test** (with an 'Edit' button)
- Male, 1960-08-28 (63), # 28081960, Other
- Uploaded By**
- Cortrium Test, Cortrium Internal > Cortrium, 2024-01-30 12:31
- 0.0% lead off
- Download recording in .EDF format (with a 'Download' button)
- Holter Reports**
- Loading data

The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The bottom left corner shows user information for 'Cortrium Test' (CT) with options to 'Edit account' and 'Log out'.

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
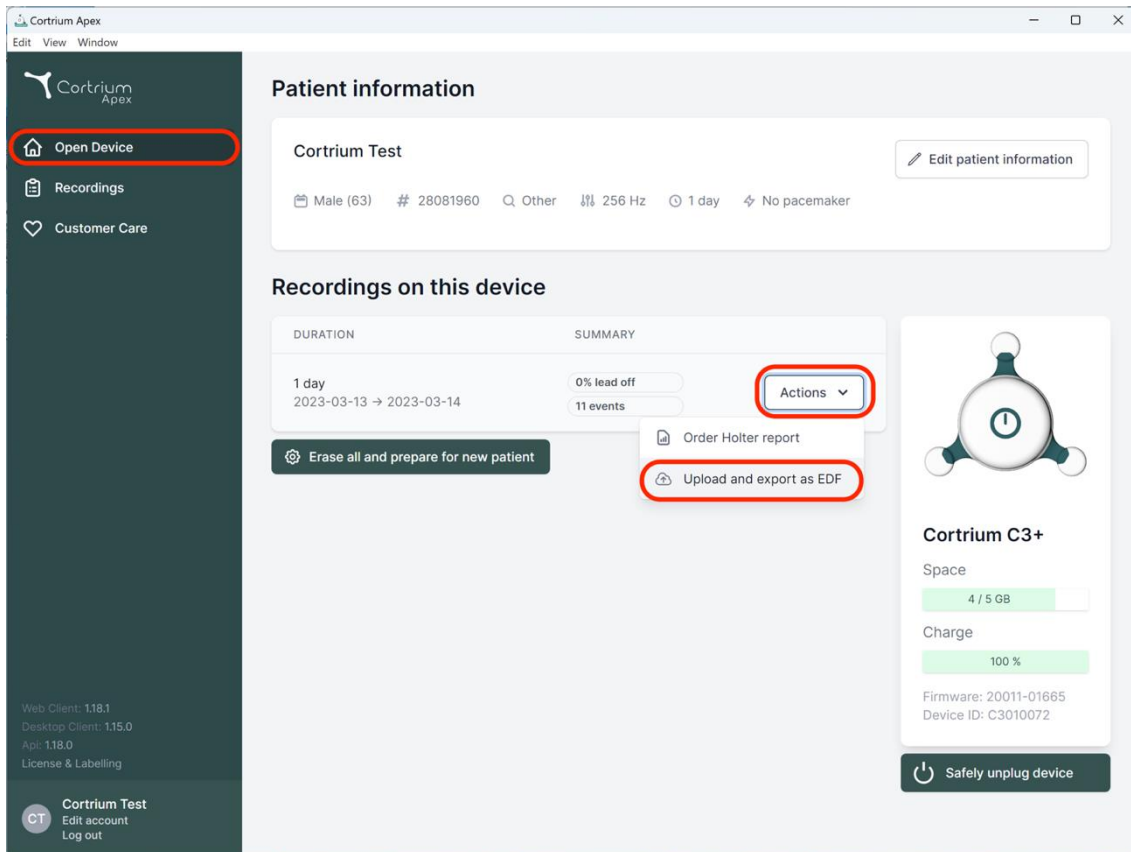
The screenshot shows the Cortrium Apex web interface. On the left, a dark sidebar contains navigation options: "Open Device" (highlighted with a red circle), "Recordings", and "Customer Care". The main content area is titled "Patient information" and shows details for "Cortrium Test", including patient demographics and test parameters. Below this, the "Recordings on this device" section displays a recording for "1 day" from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. An "Actions" dropdown menu (highlighted with a red circle) is open, showing options like "Order Holter report" and "Upload and export as EDF" (highlighted with a red circle). On the right, a "Cortrium C3+" device status card shows 4/5 GB space, 100% charge, and firmware/device ID information. A "Safely unplug device" button is at the bottom right.

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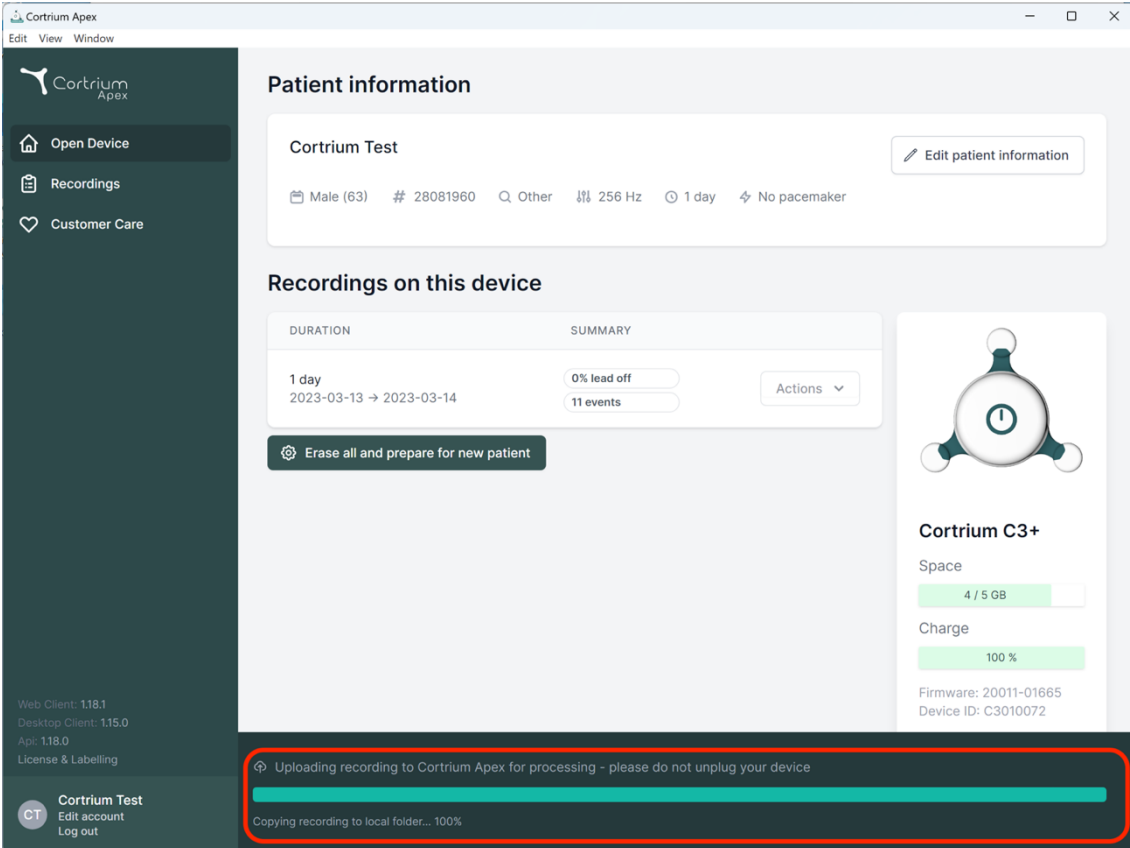Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">Actions</a>

[Erase all and prepare for new patient](#)

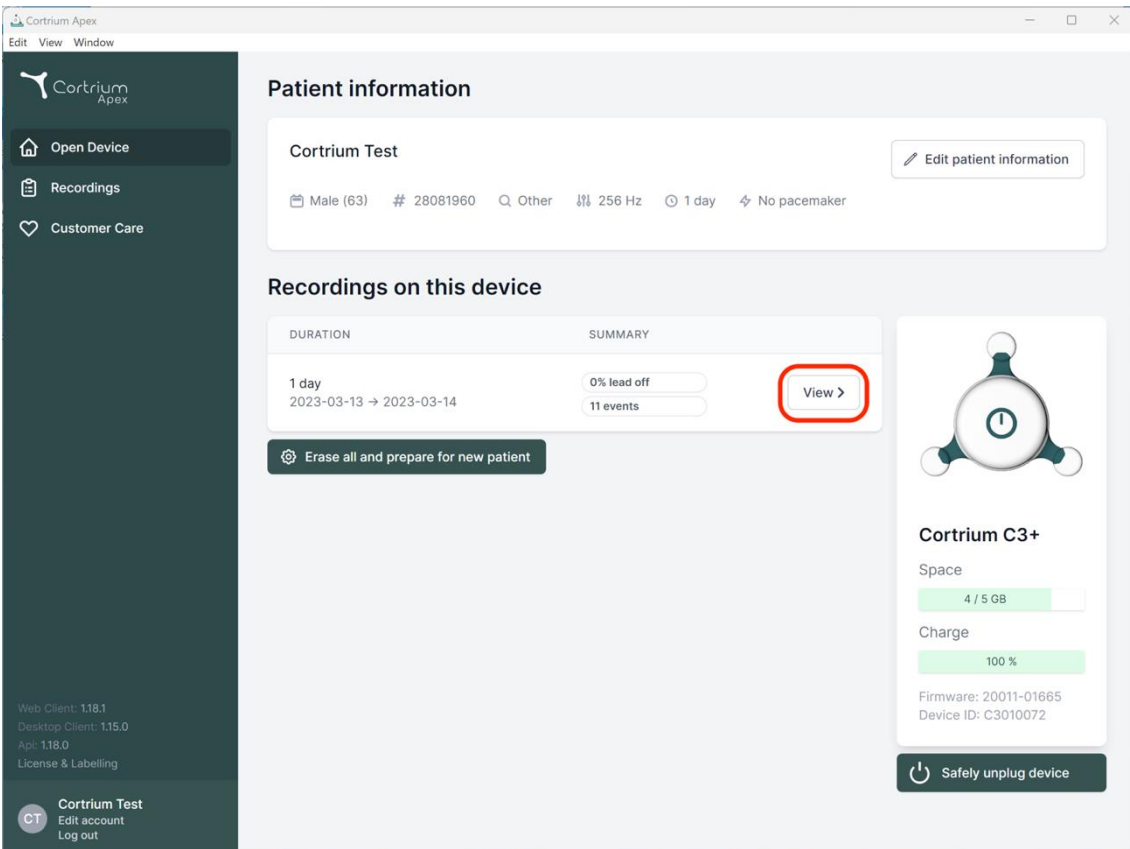
**Cortrium C3+**

Space: 4 / 5 GB  
Charge: 100 %  
Firmware: 20011-01665  
Device ID: C3010072

Web Client: 1.18.1  
Desktop Client: 1.15.0  
App: 1.18.0  
License & Labelling

**Cortrium Test**  
CT Edit account Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device  
Copying recording to local folder... 100%

**Cortrium Apex**  
Edit View Window

**Patient information**

**Cortrium Test** [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

**Recordings on this device**

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	<a href="#">View &gt;</a>

[Erase all and prepare for new patient](#)

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
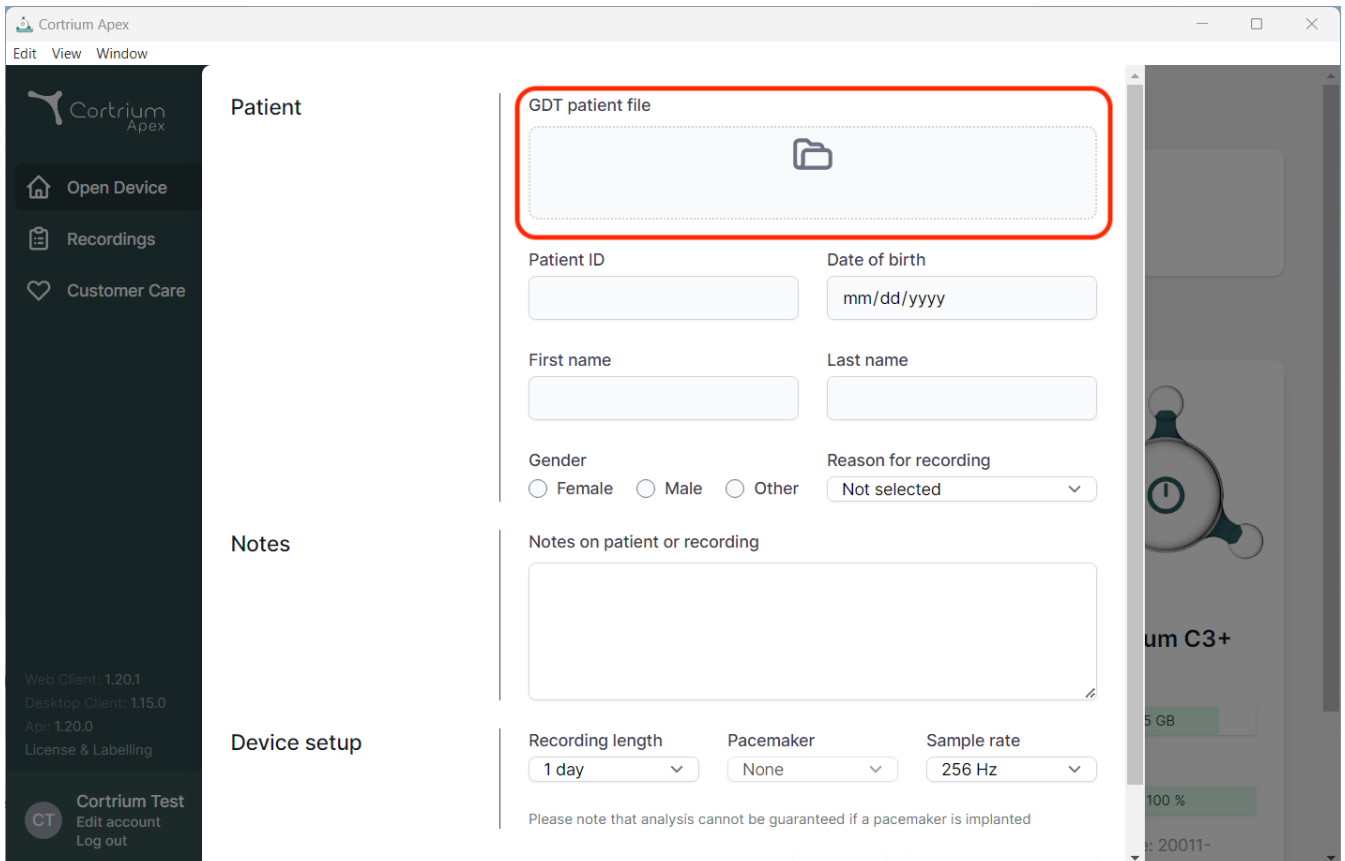
The screenshot displays the Cortrium Apex web application interface. The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section includes a red-bordered box for uploading a GDT patient file, followed by input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, Last name, Gender (Female, Male, Other), and Reason for recording (Not selected). The Notes section has a text area for notes on the patient or recording. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted".

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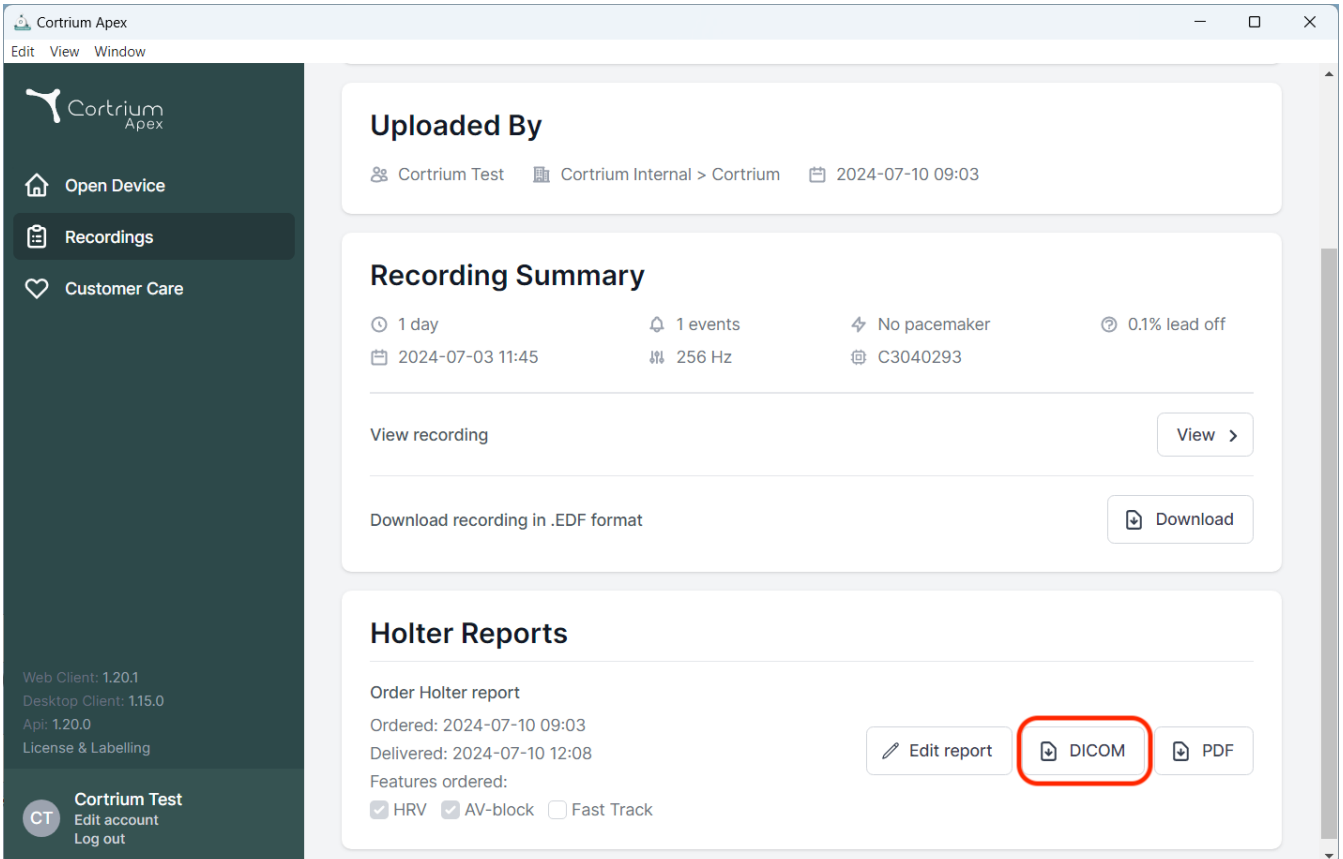The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:


- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' (2024-07-03 11:45), '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off' (C3040293). It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered' with checkboxes for 'HRV' (checked), 'AV-block' (checked), and 'Fast Track' (unchecked). Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. Version information is also visible: 'Web Client: 1.20.1', 'Desktop Client: 1.15.0', and 'Api: 1.20.0'.

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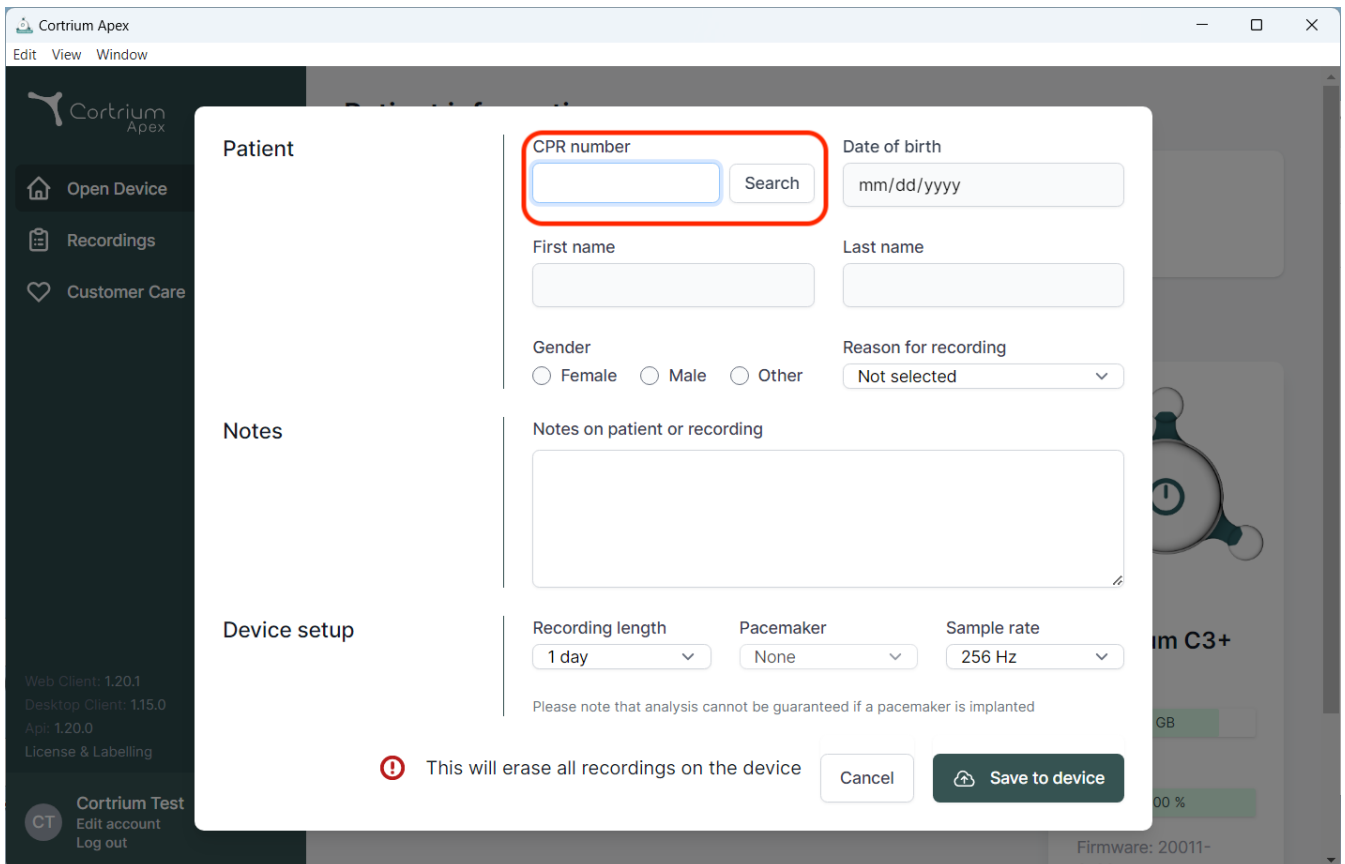Recordings

Customer Care

Web Client: 1.20.1  
Desktop Client: 1.15.0  
App: 1.20.0  
License & Labelling

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**!** This will erase all recordings on the device

Cancel

09/2024

Cortrium ApS · Erik Husfeldts Vej 7 · DK-2630 Taastrup · Denemarken  
E-mail: [info@cortrium.com](mailto:info@cortrium.com) · Website: [www.cortrium.com](http://www.cortrium.com)

**Cortrium Apex** UDI(01)05745000379033