Cortrium Apex 1.20

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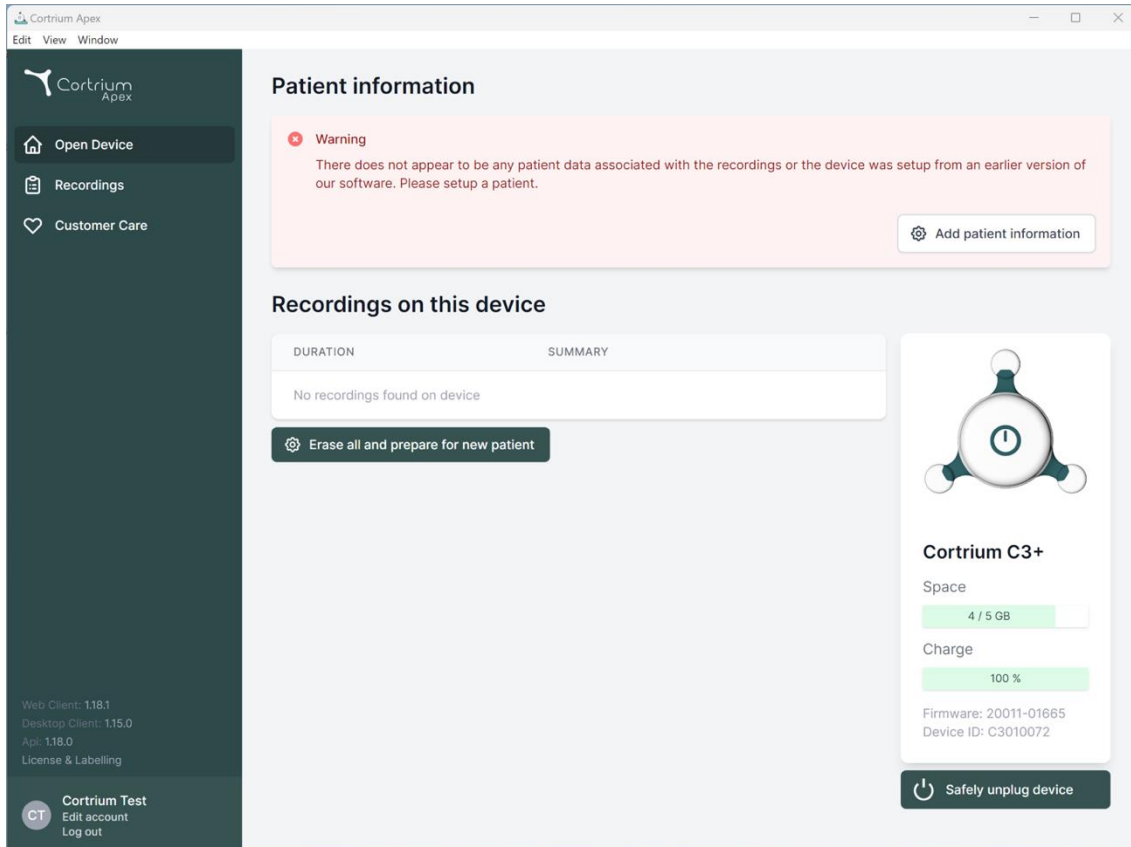
Cortrium Test
Edit account
Log out

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.

Add patient information

Recordings on this device

DURATION	SUMMARY
No recordings found on device	

Erase all and prepare for new patient

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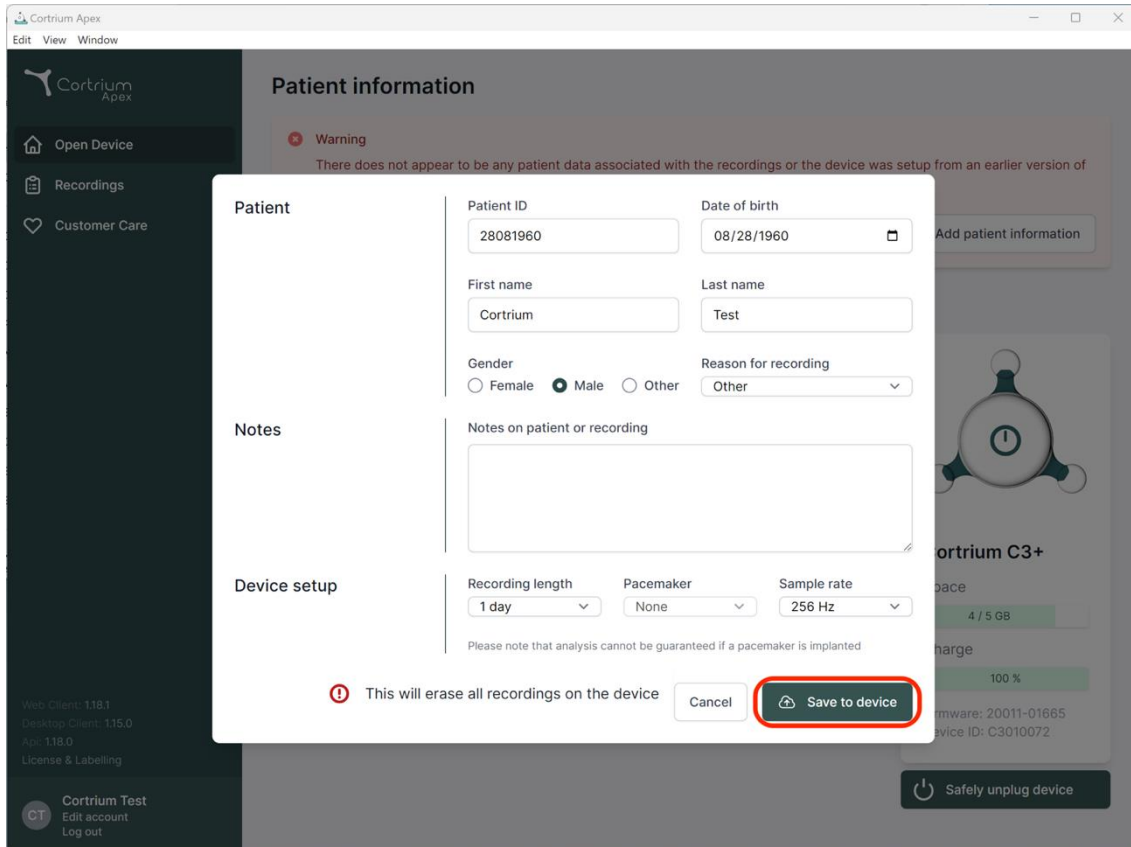
Customer Care

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: Female Male Other Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day Pacemaker: None Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

This will erase all recordings on the device Cancel **Save to device**

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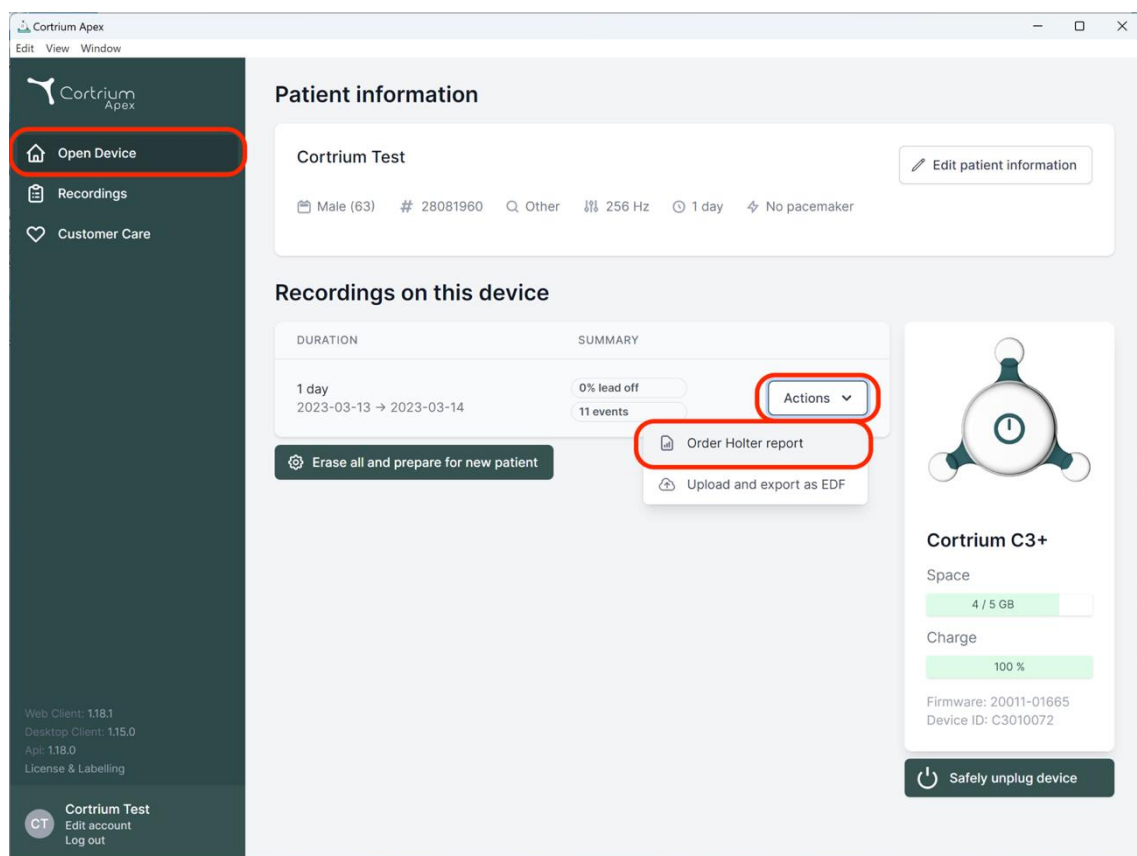
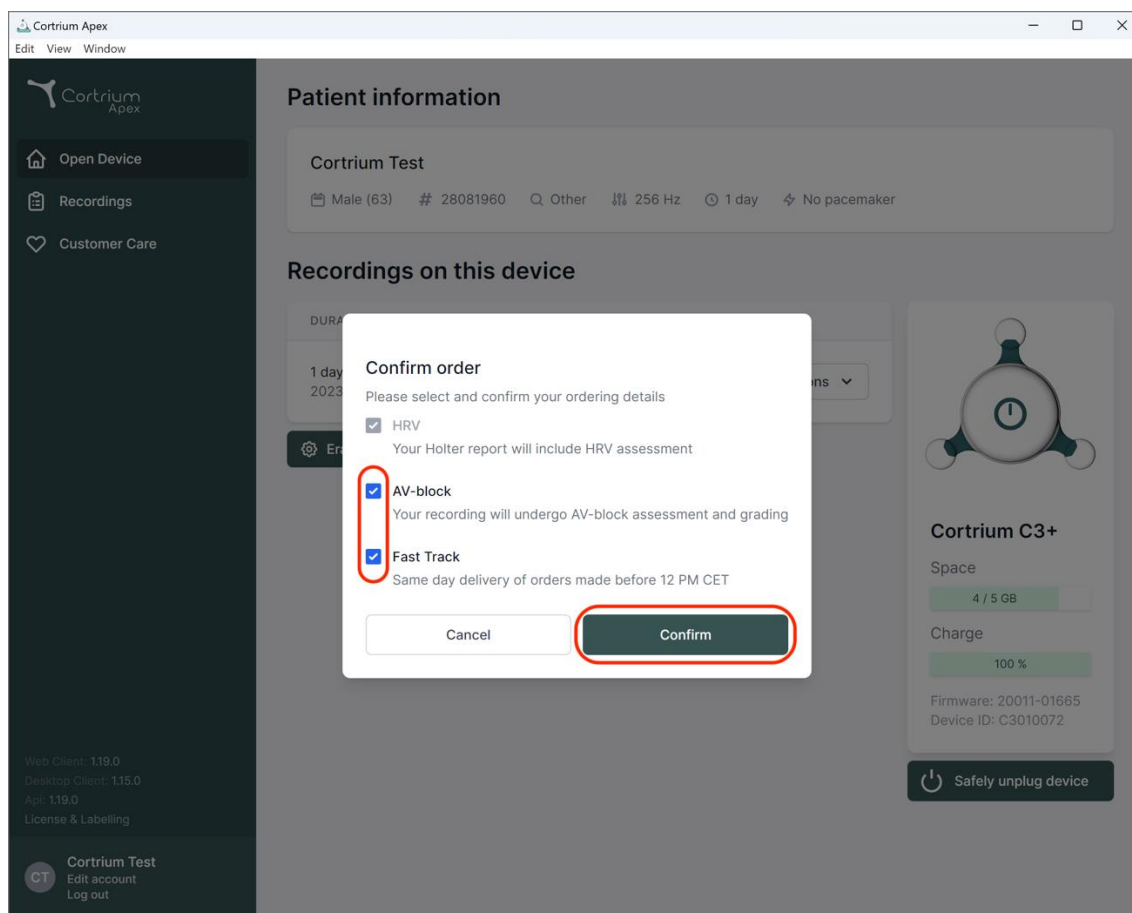
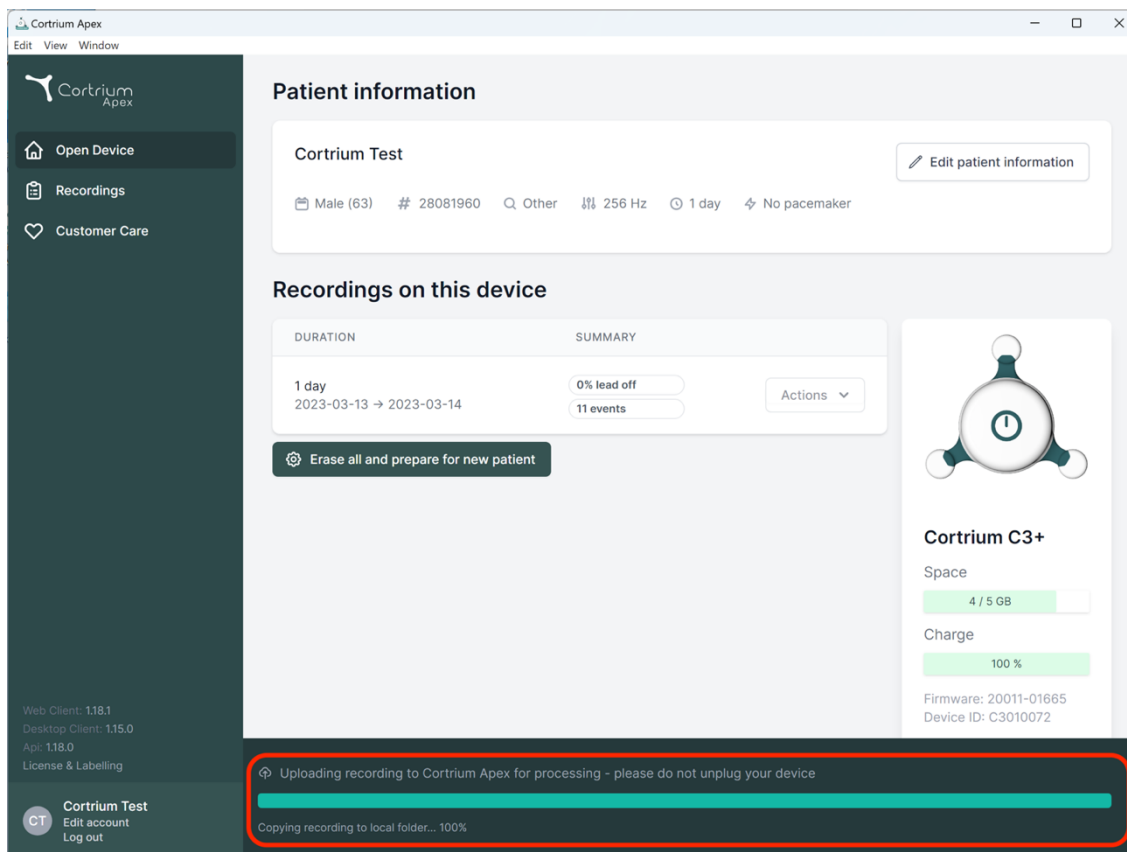
The screenshot displays the Cortrium Apex web application interface. The main content area is divided into several sections:

- Patient information:** Shows the patient name "Cortrium Test" and an "Edit patient information" button. Below this, patient details are listed: Male (63), ID # 28081960, Other, 256 Hz, 1 day, and No pacemaker.
- Recordings on this device:** A table with columns for DURATION and SUMMARY. The duration is "1 day" (2023-03-13 to 2023-03-14). The summary shows "0% lead off" and "11 events". An "Actions" dropdown menu is visible. Below the table is a button labeled "Erase all and prepare for new patient".
- Device status (Cortrium C3+):** Shows "Space: 4 / 5 GB" and "Charge: 100%". It also lists "Firmware: 20011-01665" and "Device ID: C3010072".

A red box highlights a progress bar at the bottom of the recording summary section, indicating "Uploading recording to Cortrium Apex for processing - please do not unplug your device" and "Copying recording to local folder... 100%".

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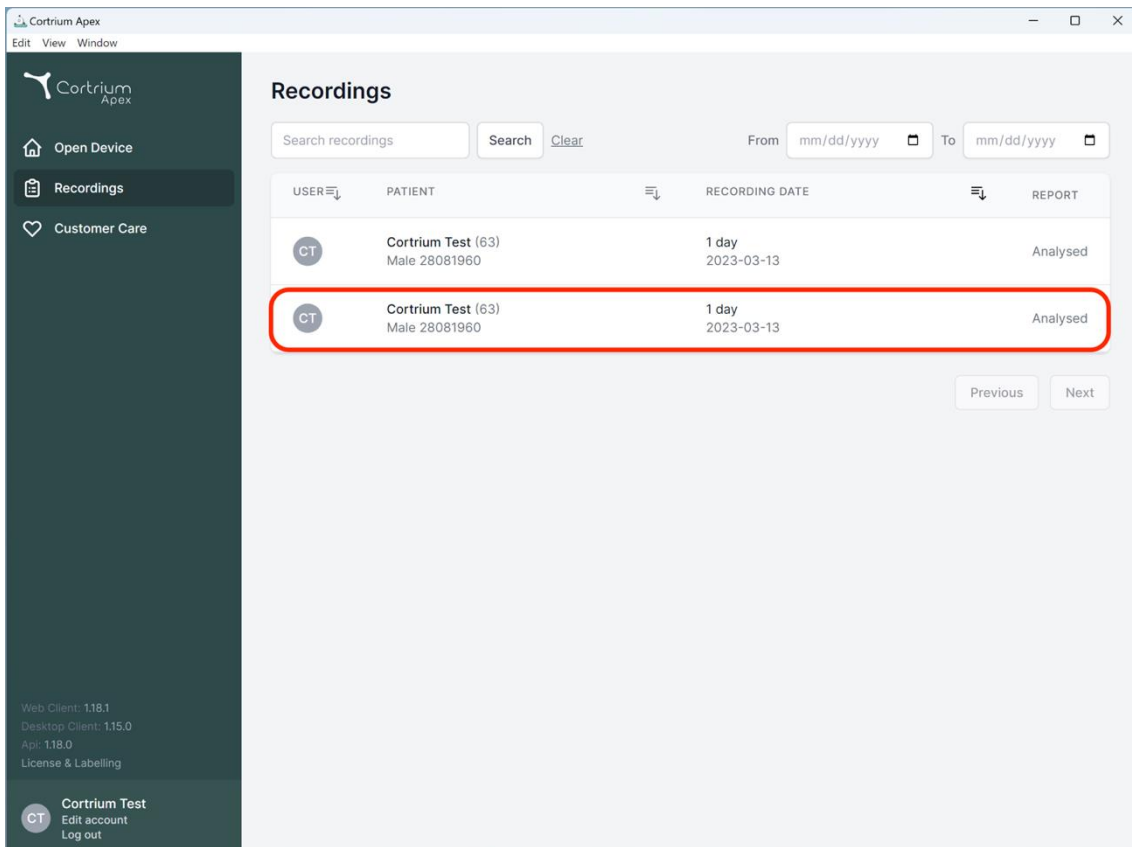
The screenshot displays the 'Recordings' section of the Cortrium Apex application. The interface includes a search bar, a date range selector, and a table of recordings. The table has the following structure:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

The second row of the table is highlighted with a red border. Below the table are 'Previous' and 'Next' navigation buttons. The left sidebar contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The footer of the sidebar lists version information: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'Api: 1.18.0', and 'License & Labelling'.

Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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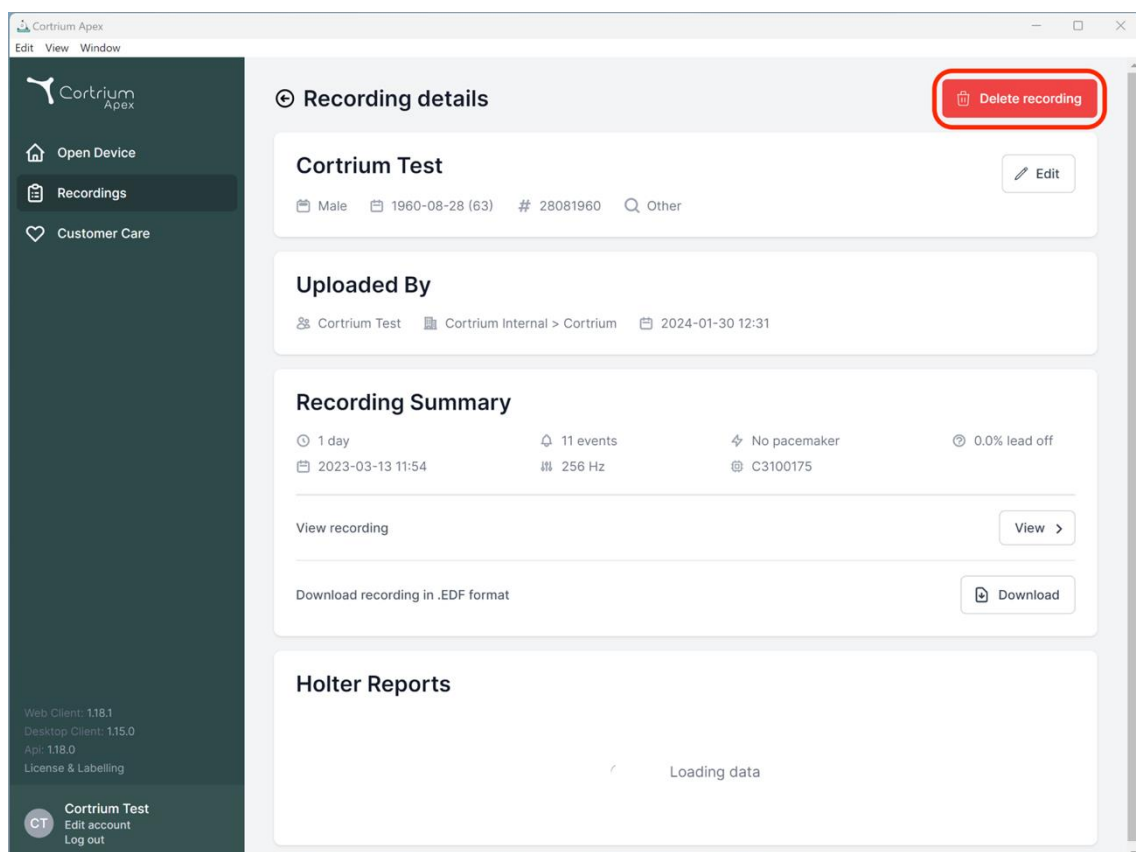
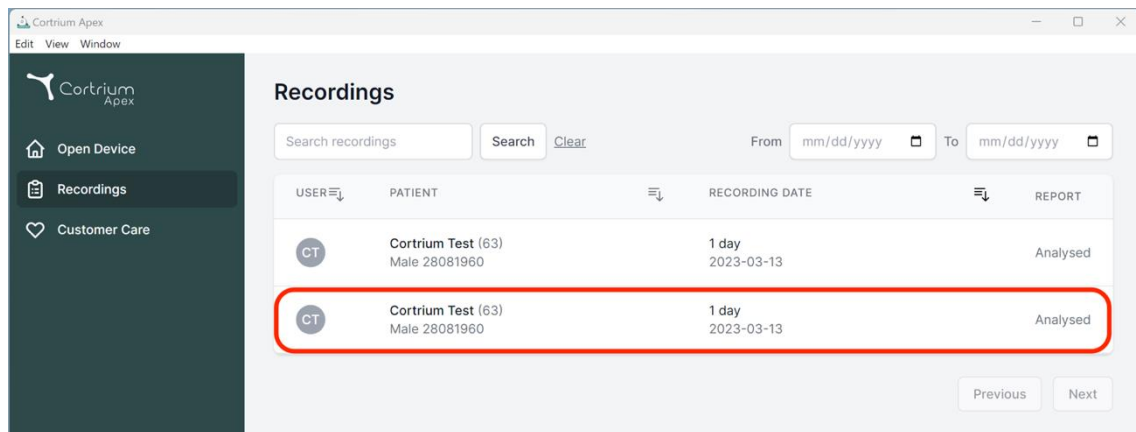
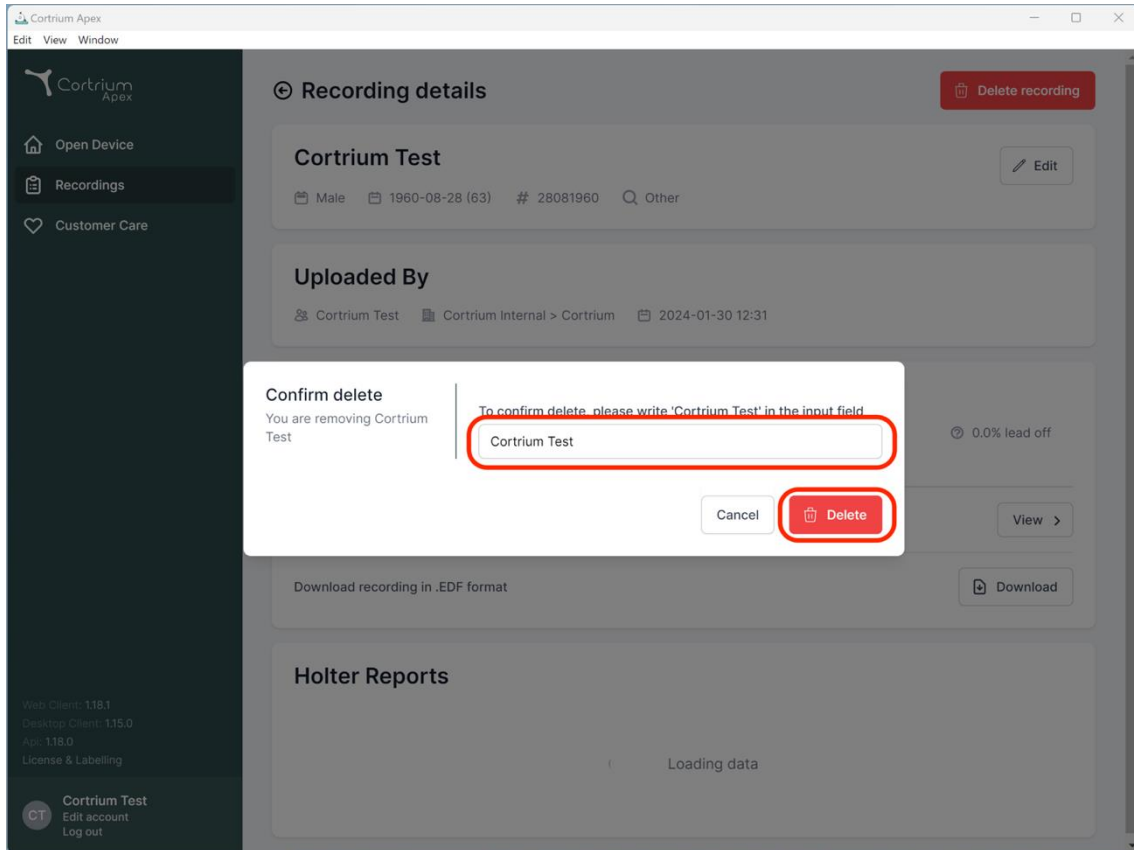
Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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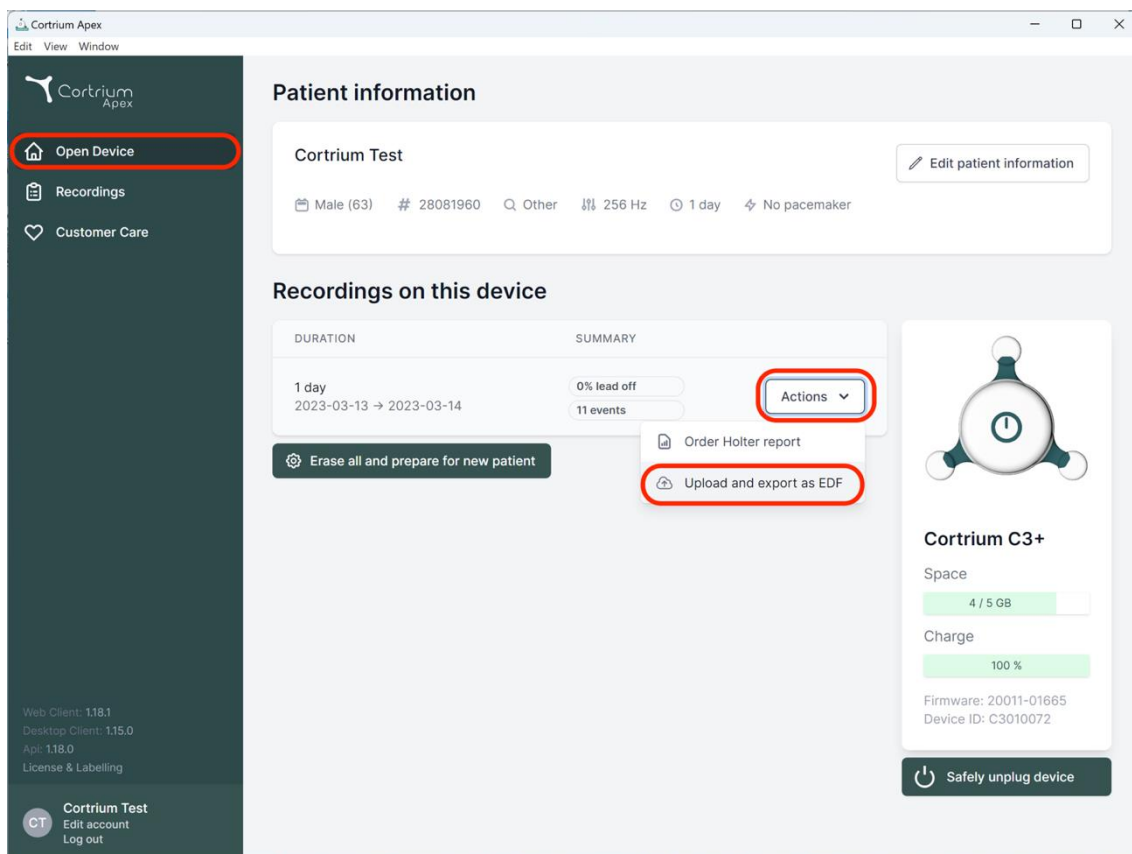
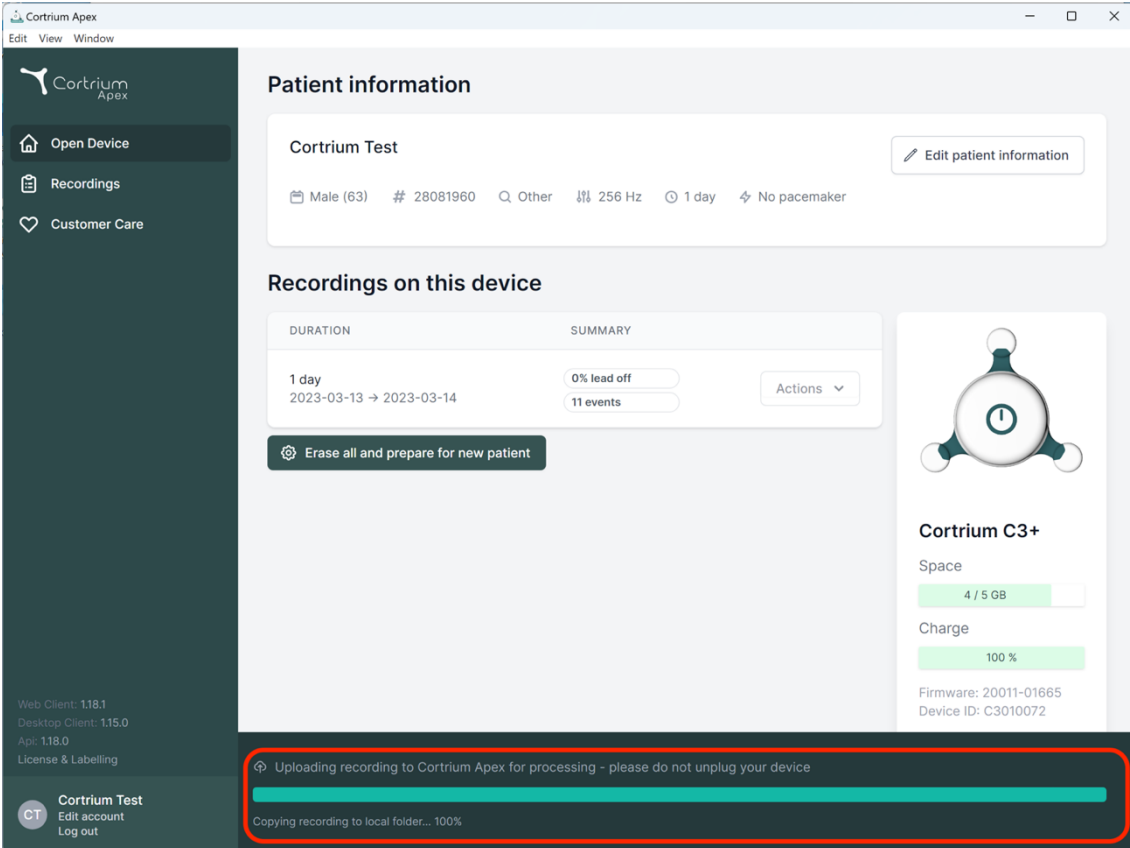
Cortrium Apex

Open Device
Recordings
Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

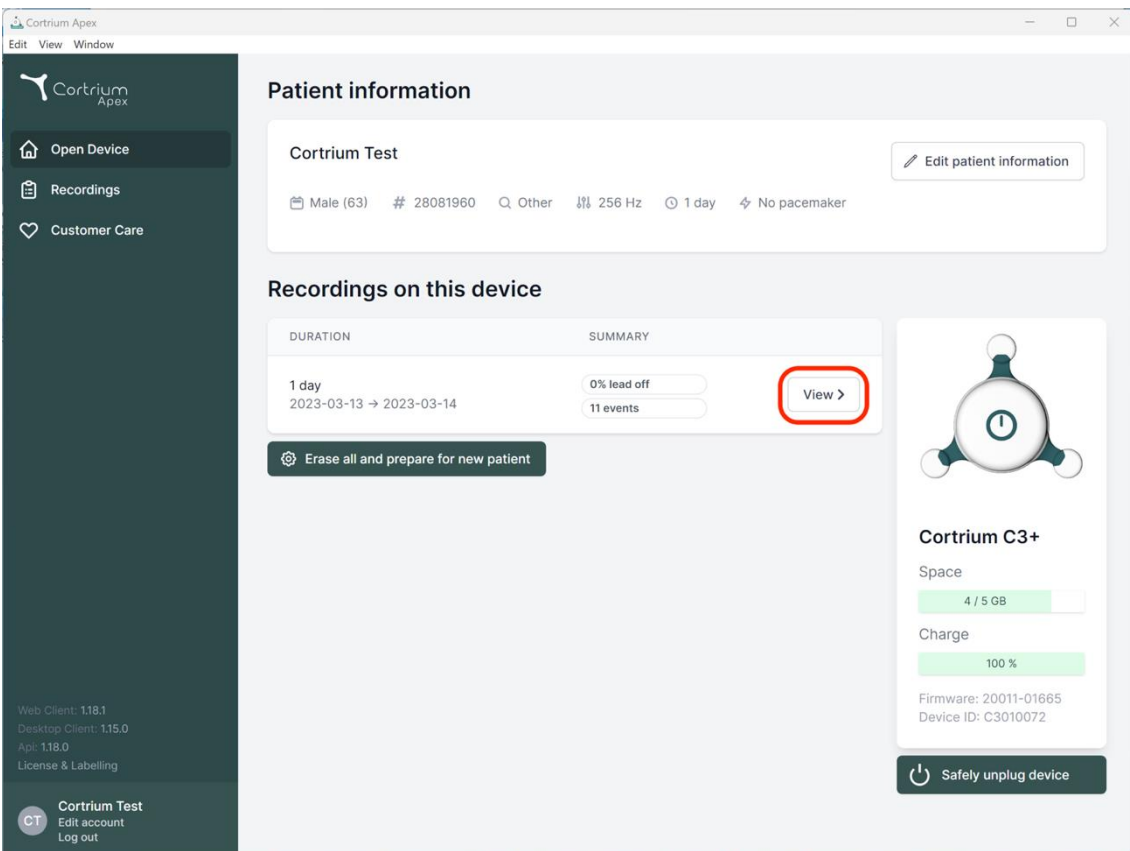
[Erase all and prepare for new patient](#)

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Cortrium C3+

Space: 4 / 5 GB
Charge: 100 %
Firmware: 20011-01665
Device ID: C3010072

Cortrium Apex

Open Device
Recordings
Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
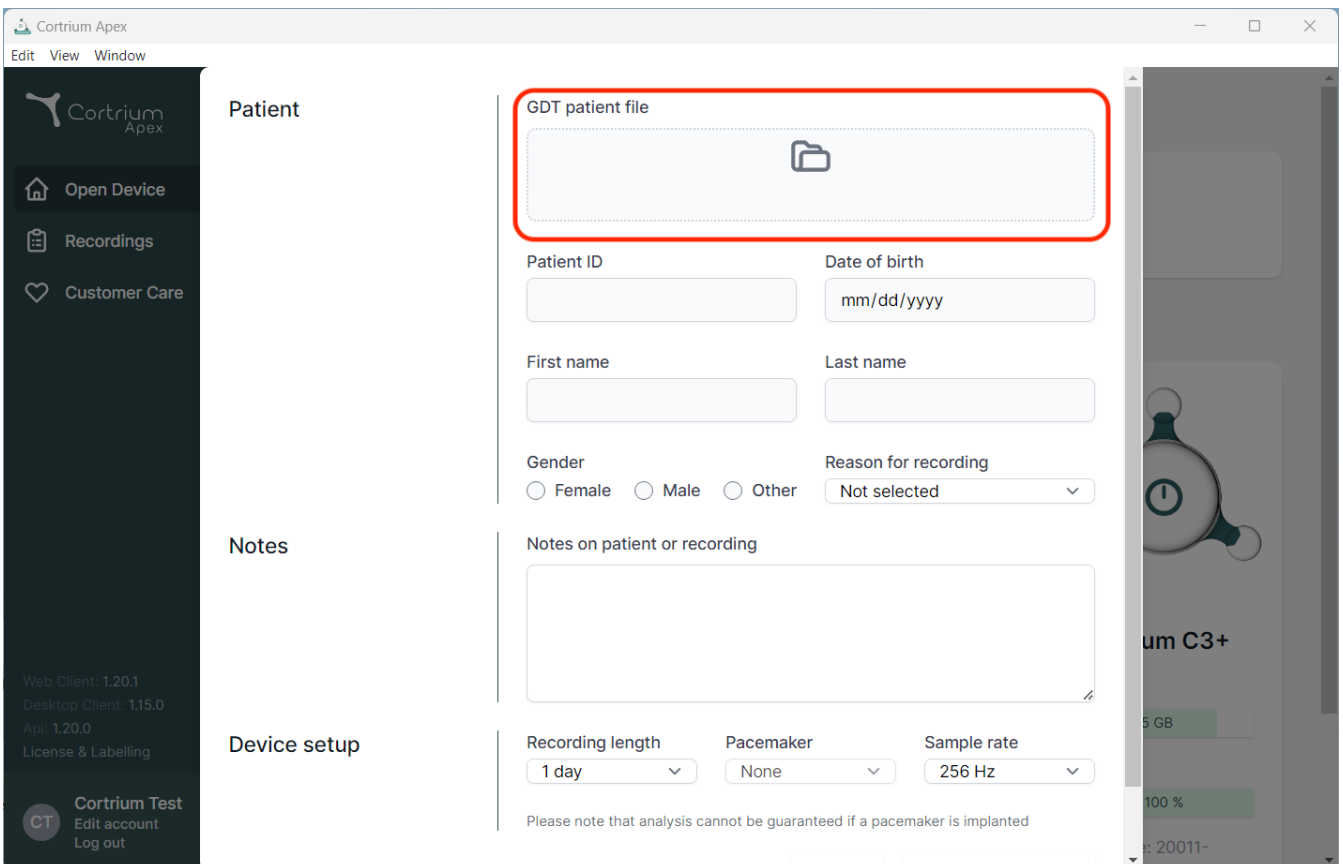
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red box highlights the "GDT patient file" field, which is a file upload area. Below this field are several input fields: "Patient ID", "Date of birth" (with a placeholder "mm/dd/yyyy"), "First name", "Last name", "Gender" (with radio buttons for Female, Male, and Other), and "Reason for recording" (a dropdown menu currently showing "Not selected"). Below these fields is a "Notes" section with a text area labeled "Notes on patient or recording". At the bottom of the form is the "Device setup" section, which includes "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below the device setup section states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options: "Open Device", "Recordings", and "Customer Care". The bottom of the sidebar shows version information: "Web Client: 1.20.1", "Desktop Client: 1.15.0", "App: 1.20.0", and "License & Labelling". The bottom of the sidebar also shows the user's name "Cortrium Test" and options to "Edit account" and "Log out".

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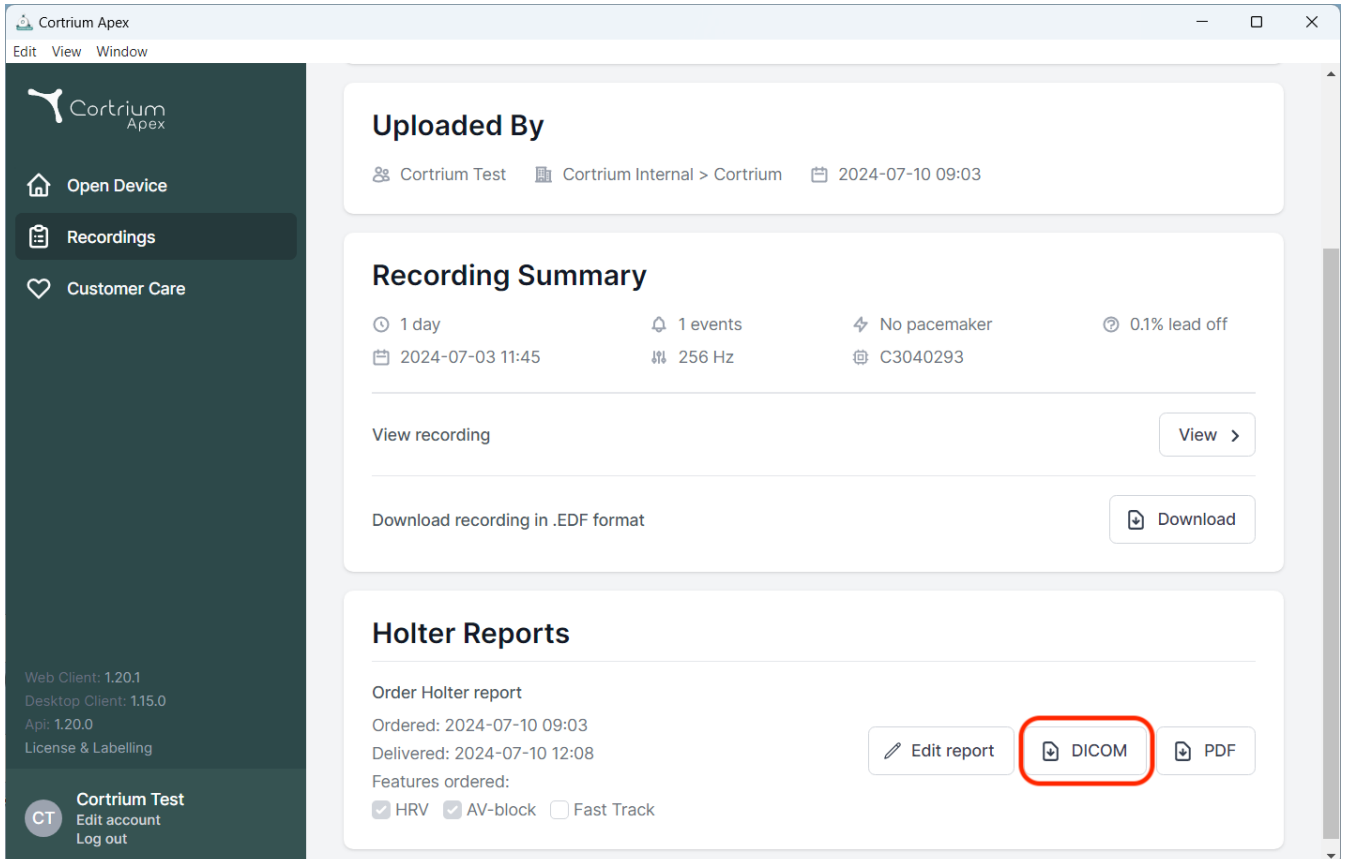
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' rate, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. Three buttons are visible: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. Version information for Web Client (1.20.1), Desktop Client (1.15.0), and Api (1.20.0) is also present.

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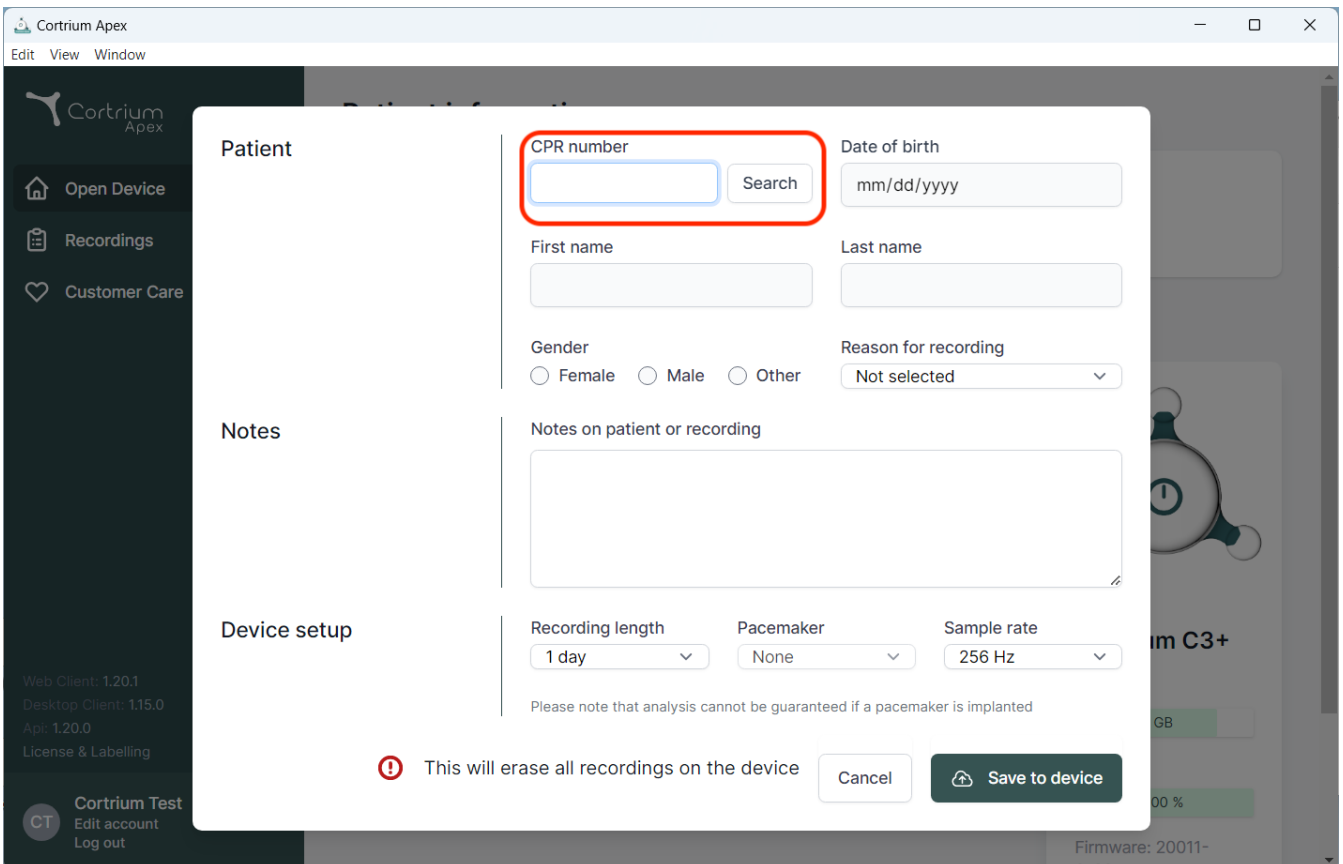The screenshot displays the Cortrium Apex software interface. A modal window titled "Patient" is open, containing the following fields and controls:

- CPR number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangular box.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup fields, a warning message is displayed: **!** This will erase all recordings on the device. At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

09/2024

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dania
E-mail: info@cortrium.com • Strona internetowa: www.cortrium.com

Cortrium Apex UDI(01)05745000379033