Cortrium Apex 1.20

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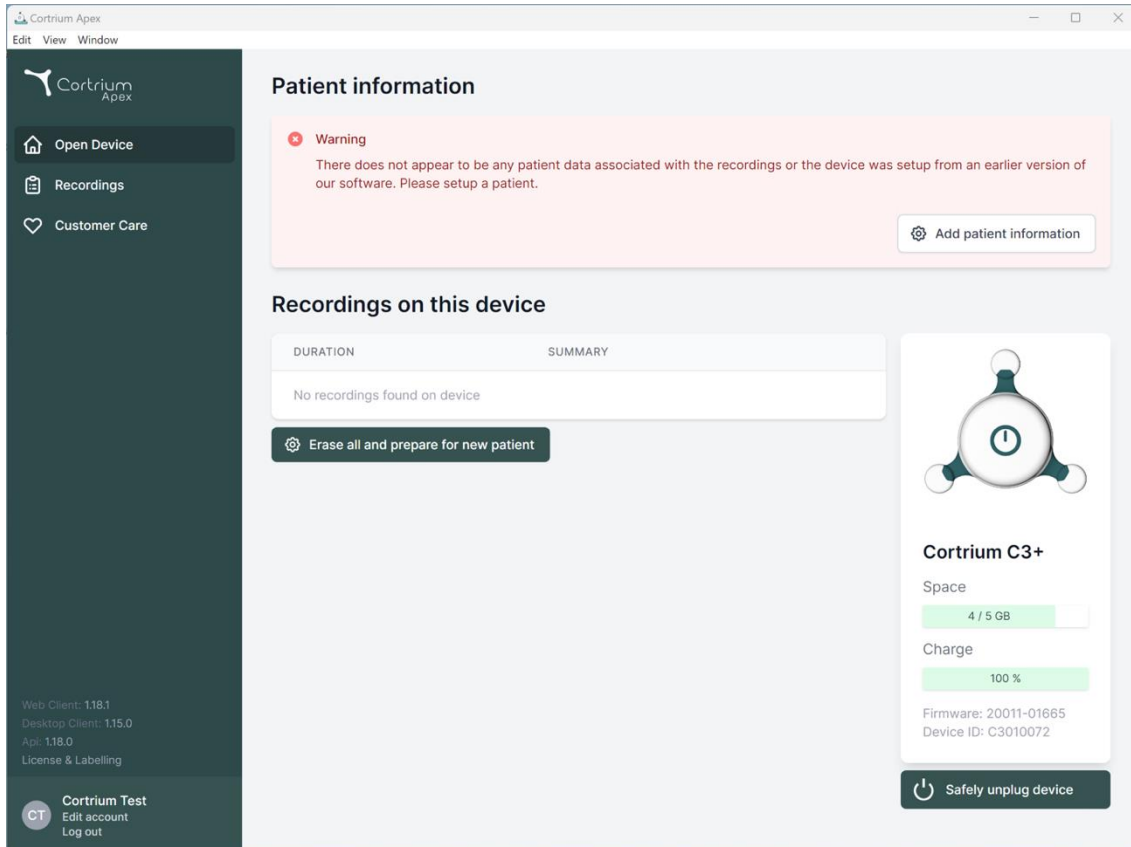
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0) and a 'License & Labelling' link. At the bottom of the sidebar is a 'Cortrium Test' section with 'Edit account' and 'Log out' options. The main content area is titled 'Patient information' and features a warning message: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table header for 'DURATION' and 'SUMMARY', containing the text 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' shows 'Space' at 4 / 5 GB, 'Charge' at 100%, and 'Firmware: 20011-01665 Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex
Edit View Window

Cortrium Apex

Open Device
Recordings
Customer Care

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: Female Male Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

⚠ This will erase all recordings on the device

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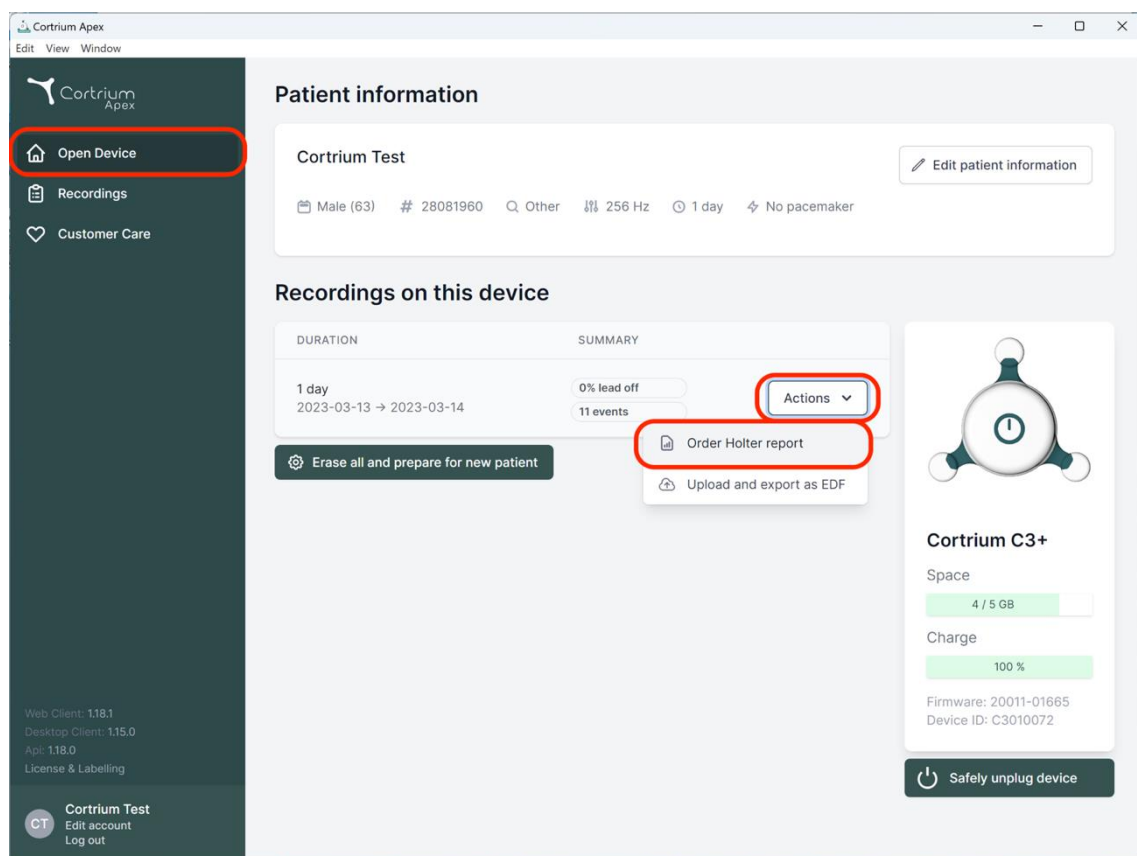
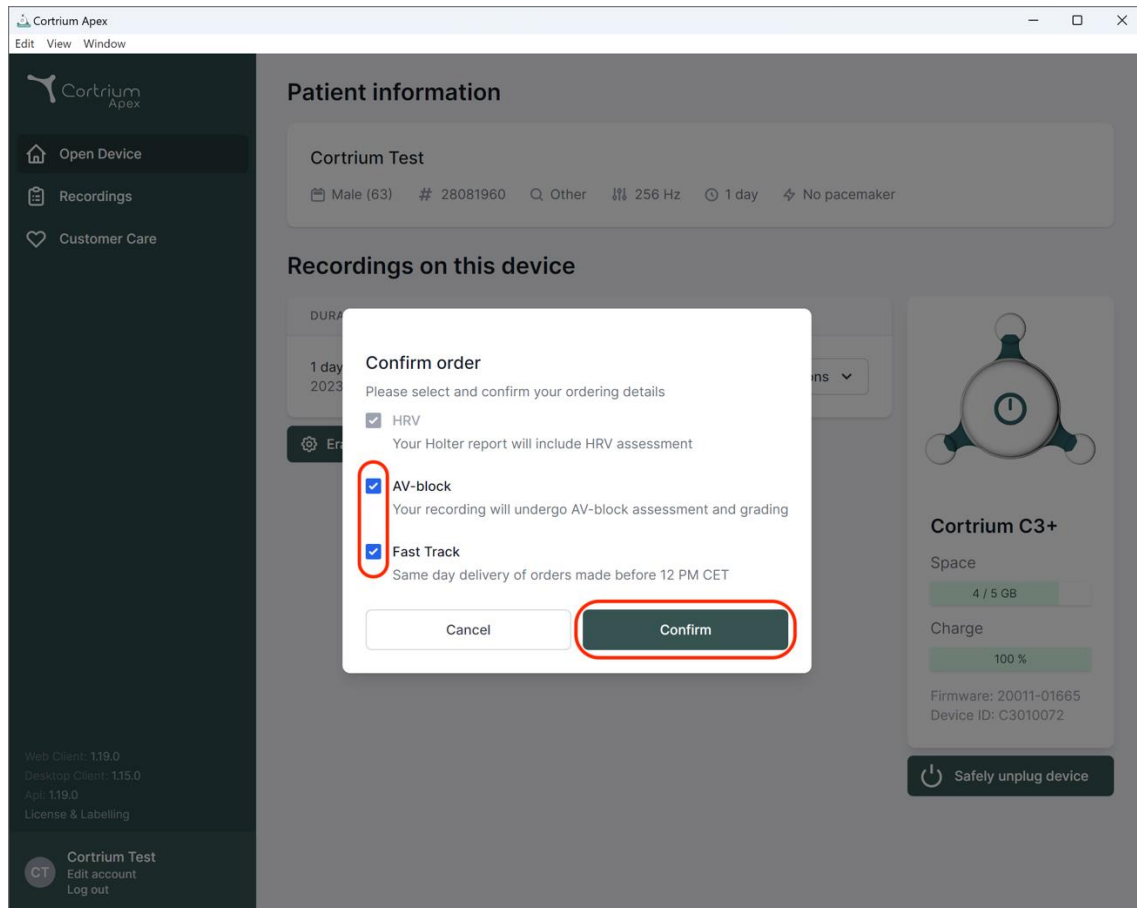
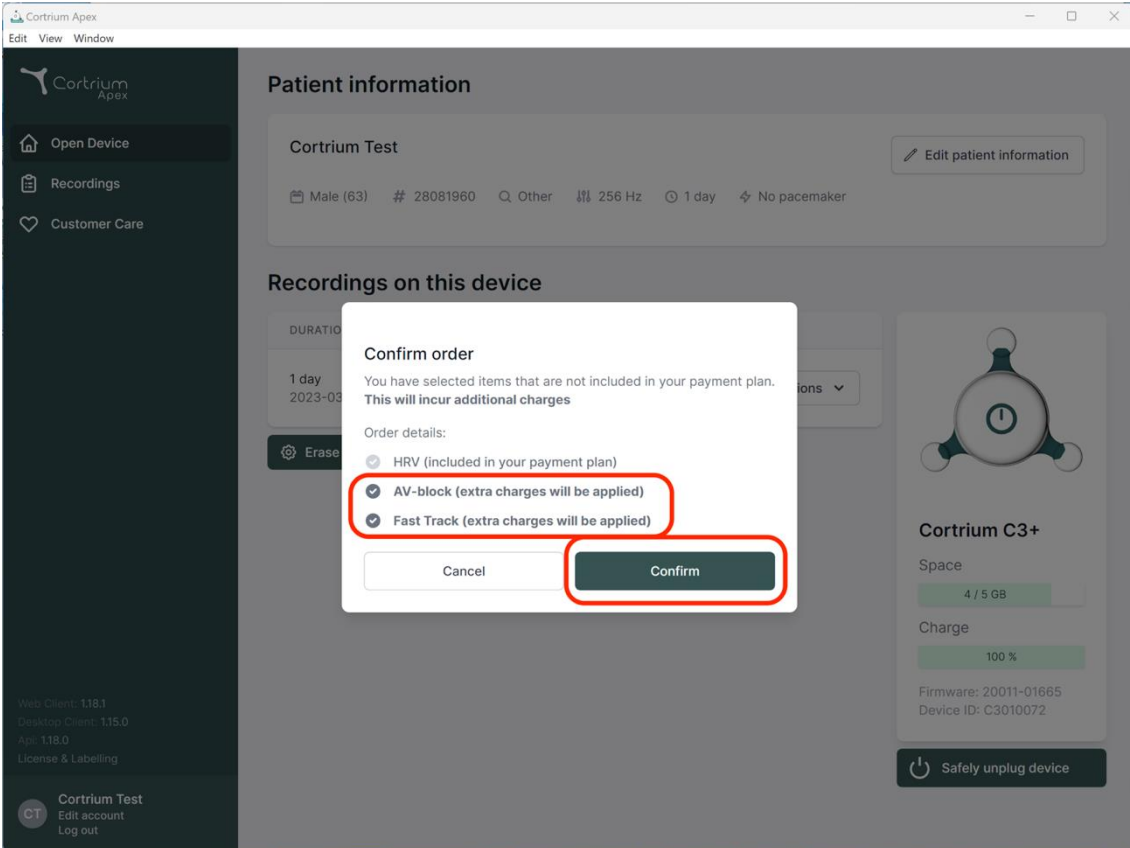
Erase

Confirm order

You have selected items that are not included in your payment plan.
This will incur additional charges

Order details:

- HRV (included in your payment plan)
- AV-block (extra charges will be applied)
- Fast Track (extra charges will be applied)

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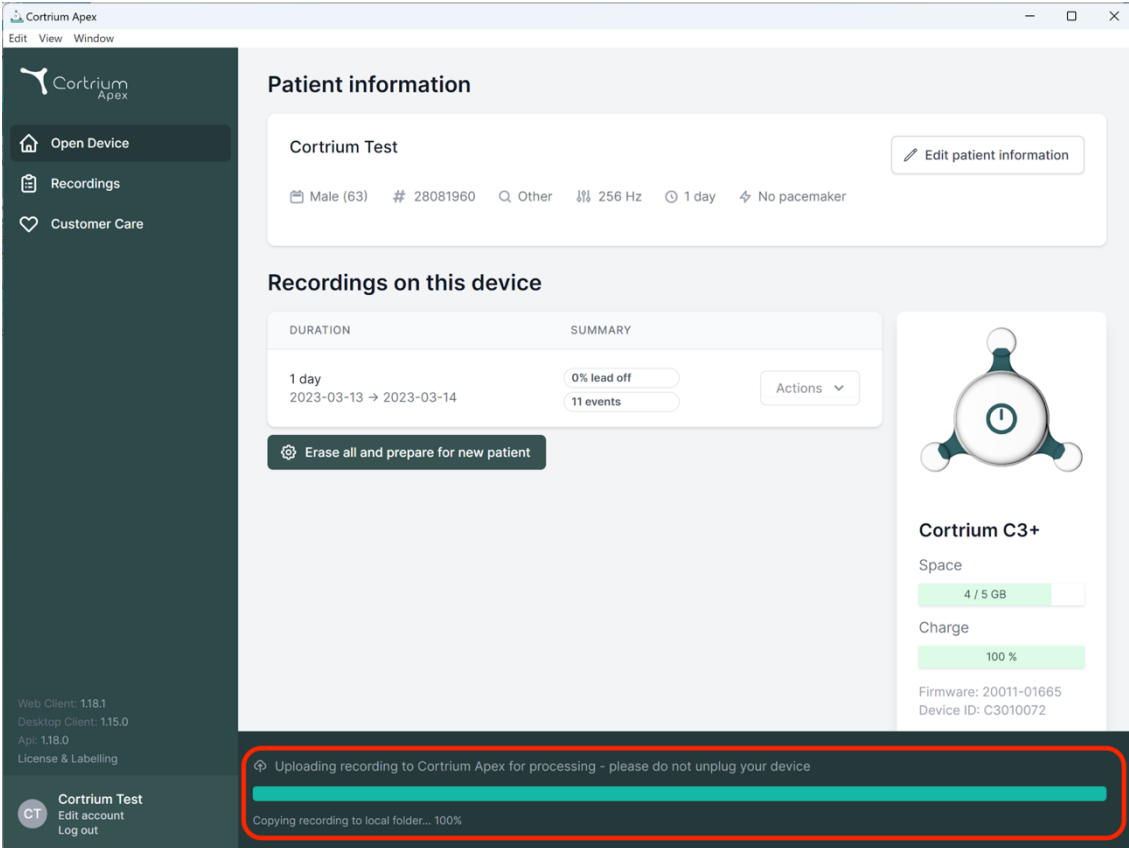
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, the 'Recordings on this device' section shows a summary of 0% lead off and 11 events. A 'Cortrium C3+' device status card on the right indicates 4/5 GB of space used and 100% charge. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device', with a sub-progress for 'Copying recording to local folder... 100%'.

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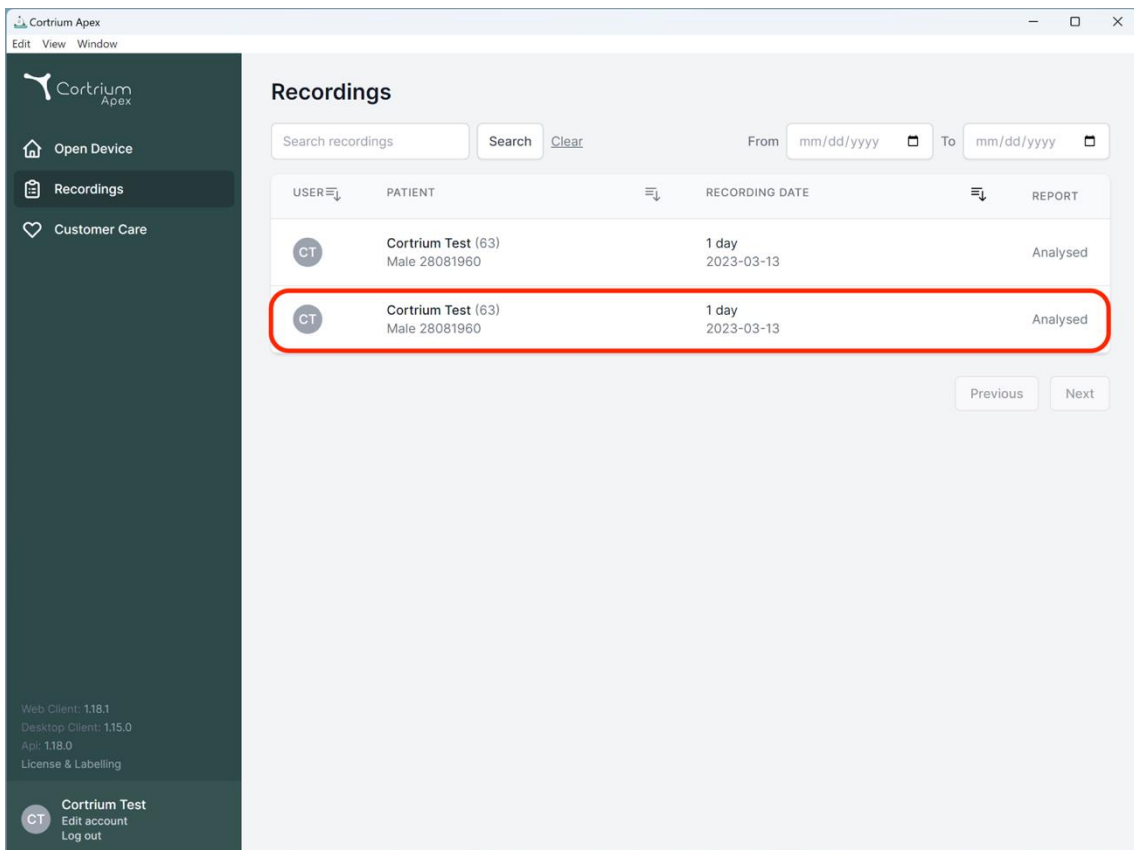
The screenshot shows the Cortrium Apex web application interface. The left sidebar contains navigation options: Open Device, Recordings (selected), and Customer Care. The main content area is titled "Recordings" and features a search bar, a "Search" button, and a "Clear" link. Below the search bar is a table with the following columns: USER, PATIENT, RECORDING DATE, and REPORT. The table contains two rows of data, with the second row highlighted by a red border. The bottom of the sidebar shows version information and user options.

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

Web Client: 1.18.1
Desktop Client: 1.15.0
Api: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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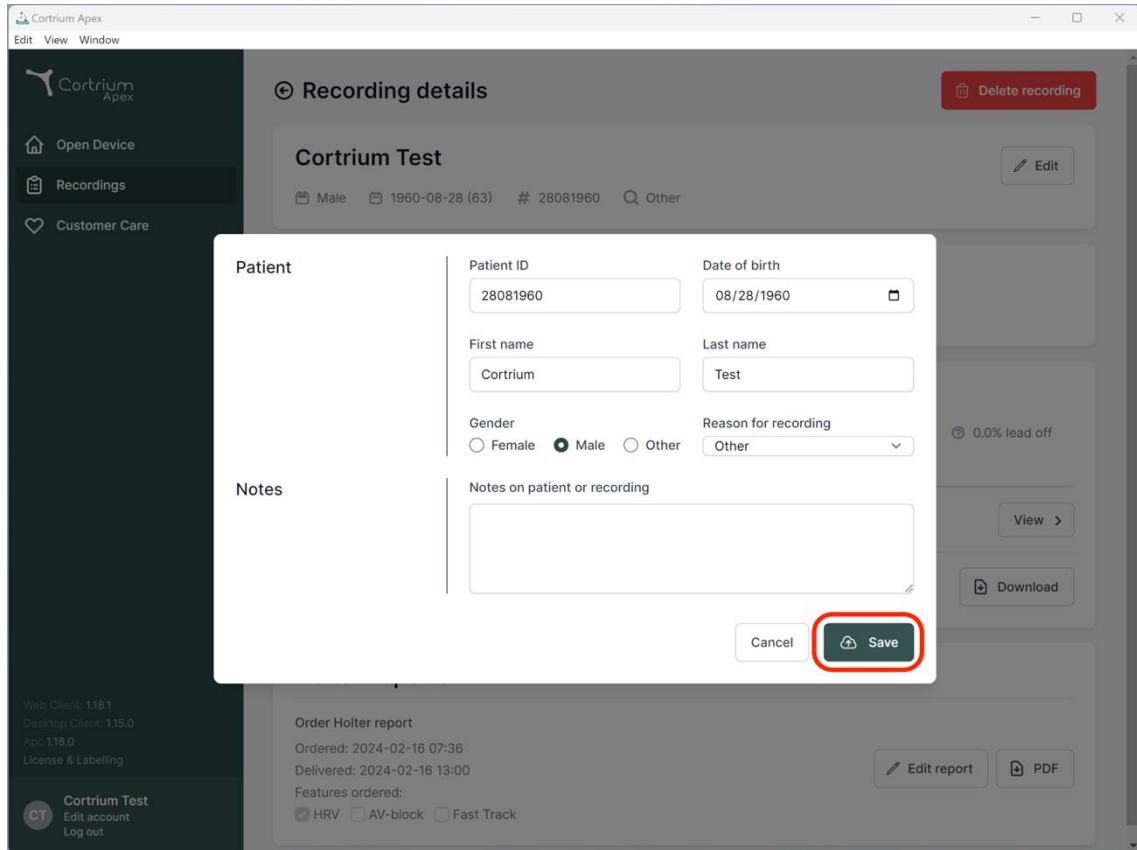
Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: Female Male Other Reason for recording: Other

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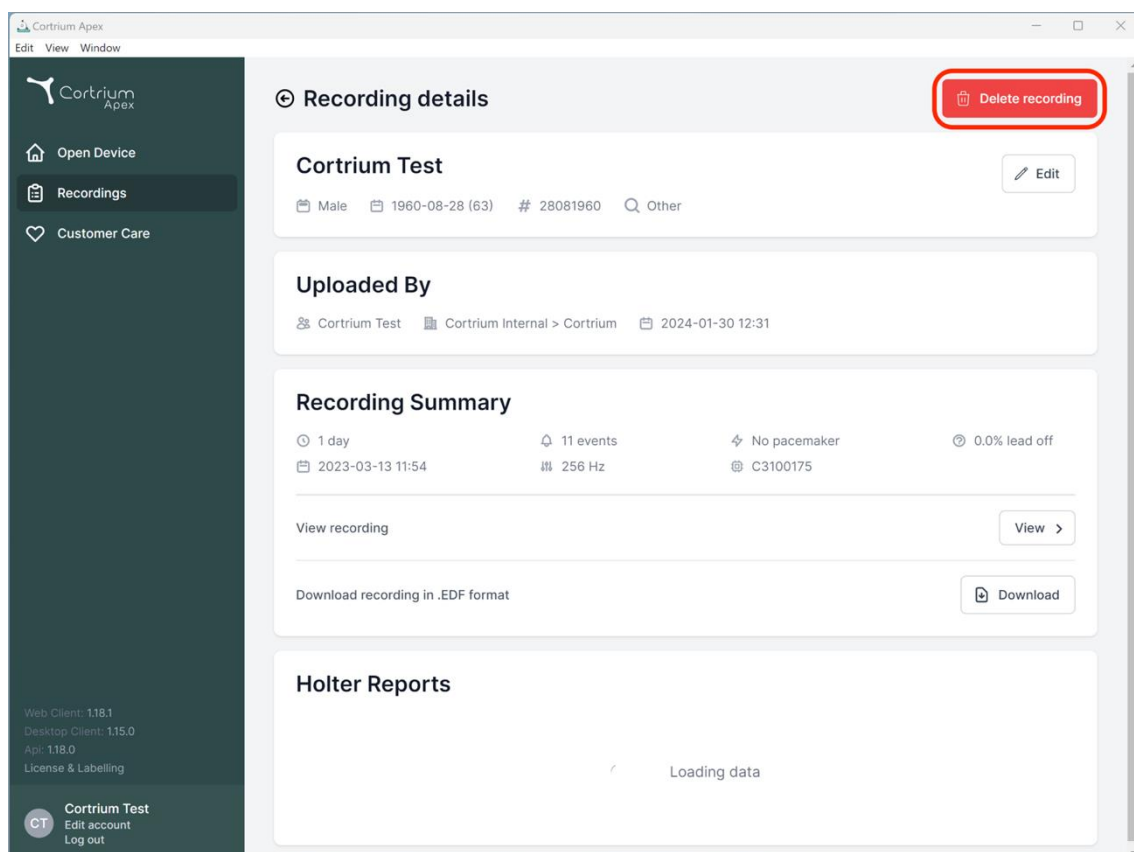
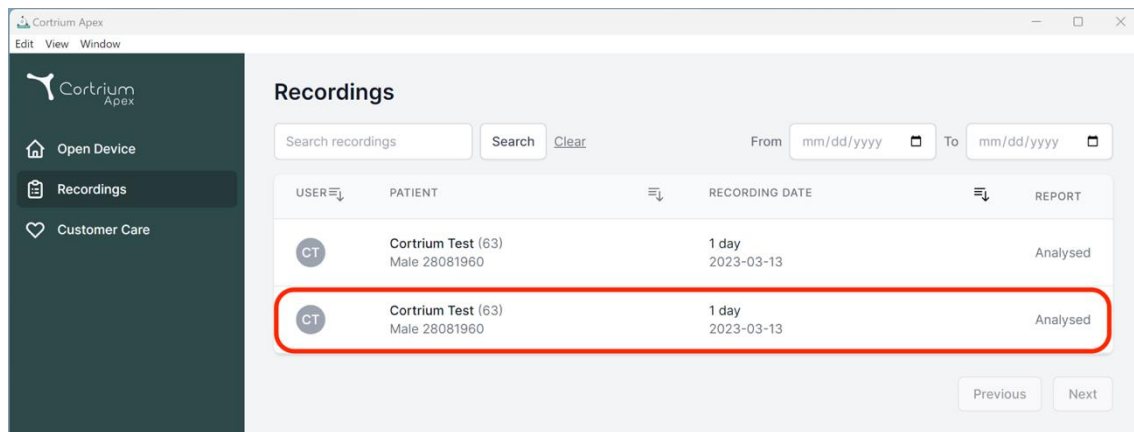
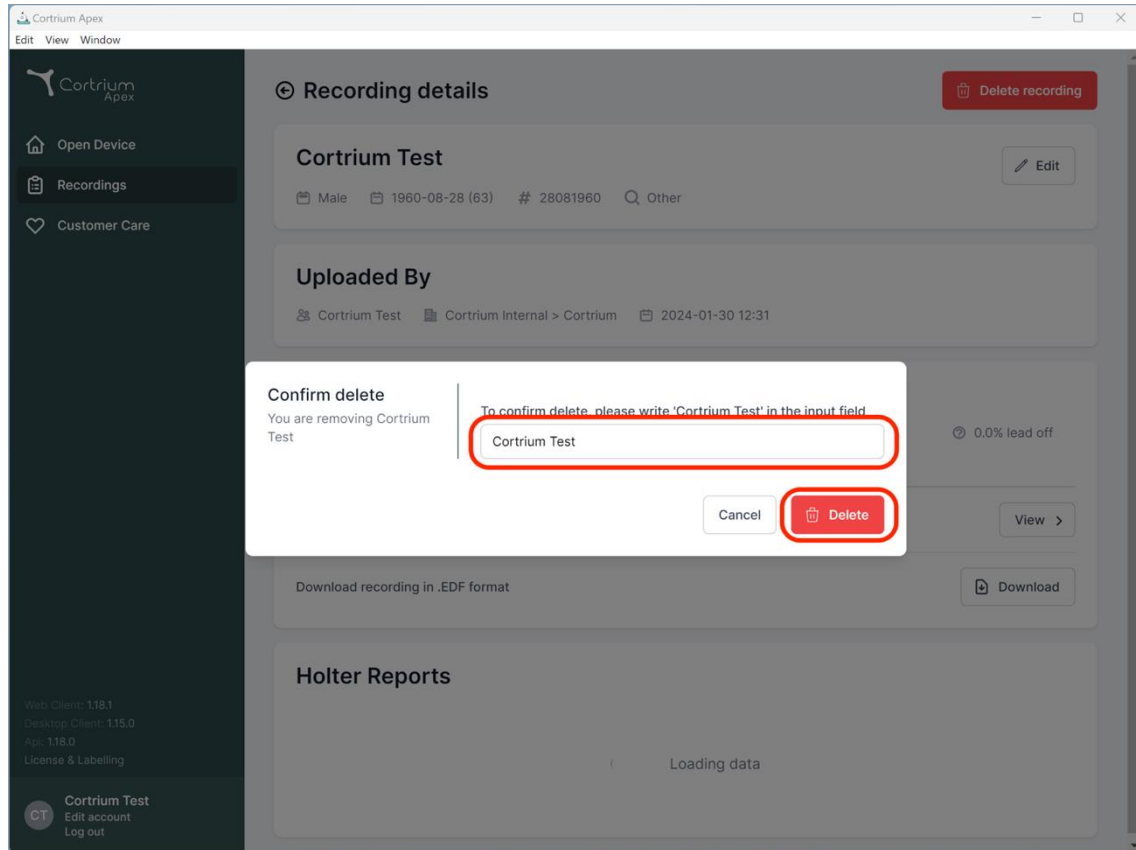
The screenshot shows the Cortrium Apex web application interface. A 'Confirm delete' dialog box is open in the foreground, overlaid on the 'Recording details' page. The dialog box contains the following text and elements:

- Confirm delete**
- You are removing Cortrium Test
- To confirm delete, please write 'Cortrium Test' in the input field.
- An input field containing the text 'Cortrium Test'.
- Buttons for 'Cancel' and 'Delete'.

The background page, 'Recording details', shows the following information:

- Recording details** (with a 'Delete recording' button)
- Cortrium Test** (with an 'Edit' button)
- Male, 1960-08-28 (63), # 28081960, Other
- Uploaded By**
- Cortrium Test, Cortrium Internal > Cortrium, 2024-01-30 12:31
- 0.0% lead off
- Download recording in .EDF format (with a 'Download' button)
- Holter Reports**
- Loading data

The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. At the bottom of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The top of the browser window shows 'Cortrium Apex' and standard window controls.

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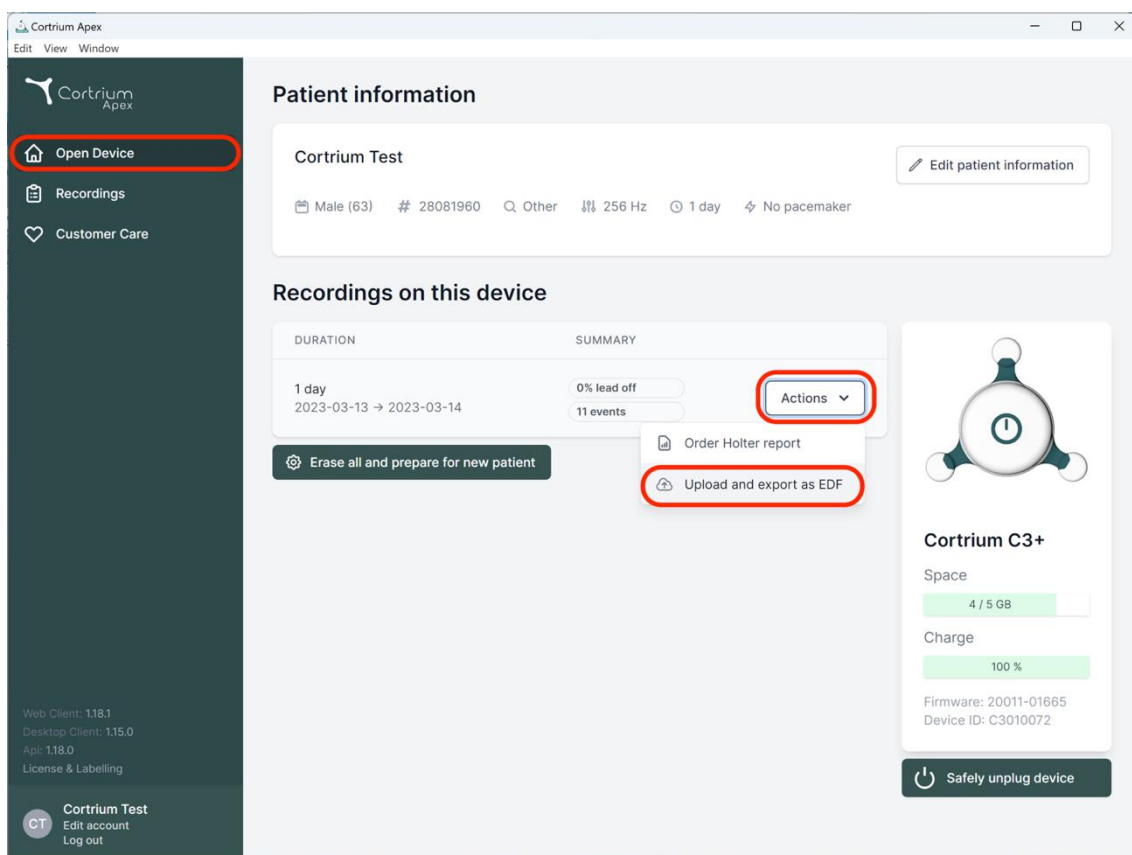
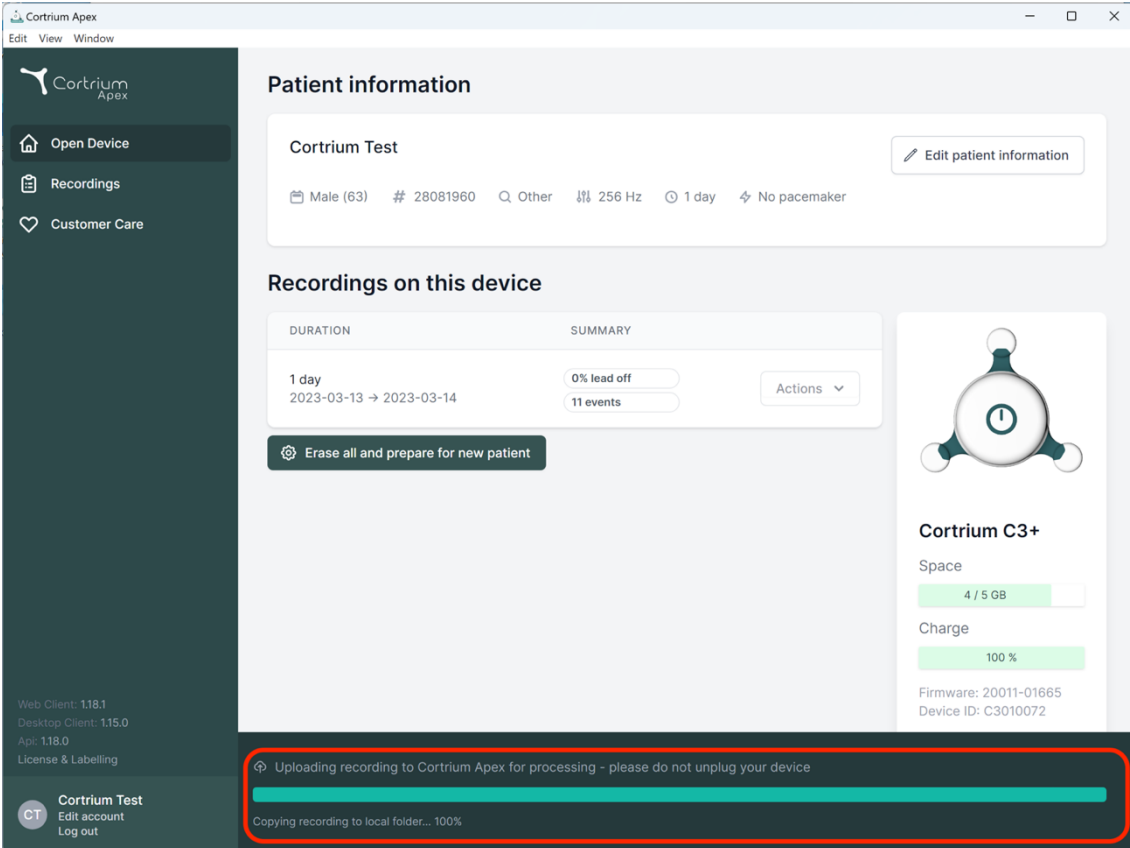
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

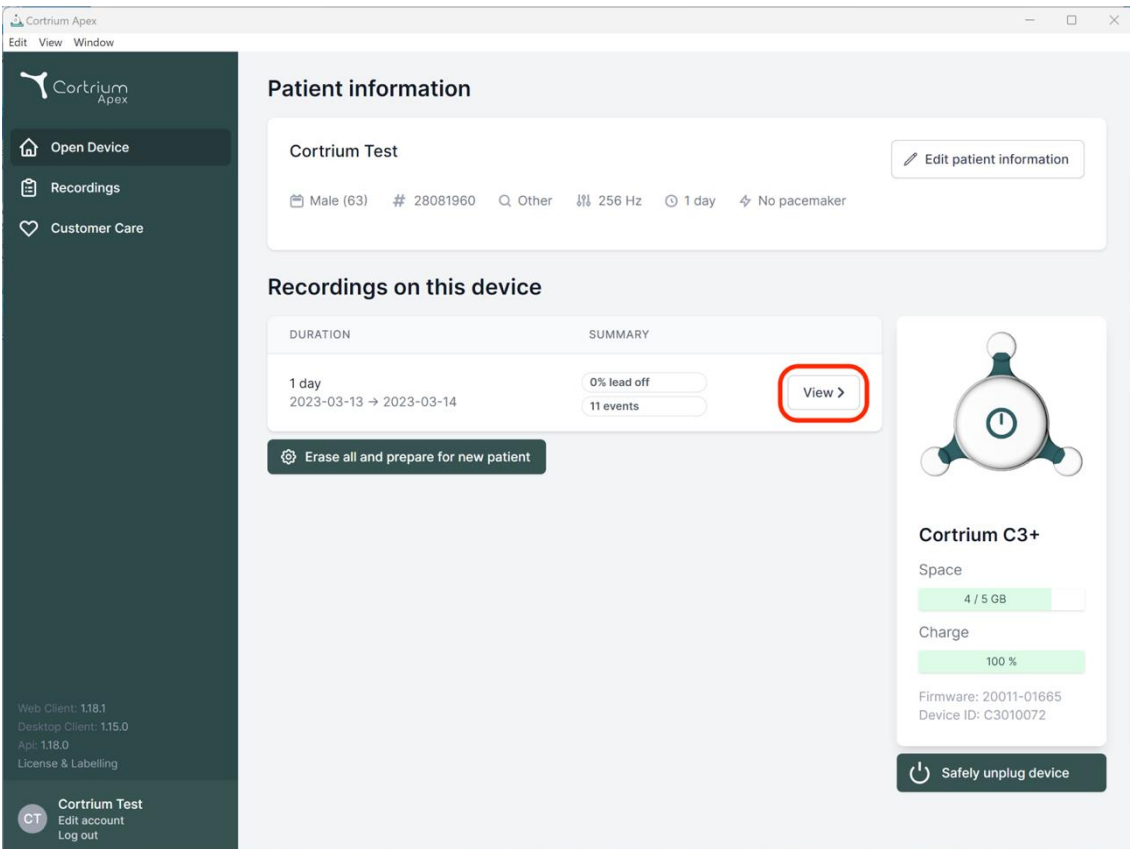
Firmware: 20011-01665
Device ID: C3010072

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

Cortrium Test
CT Edit account Log out

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Cortrium Apex

Edit View Window

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
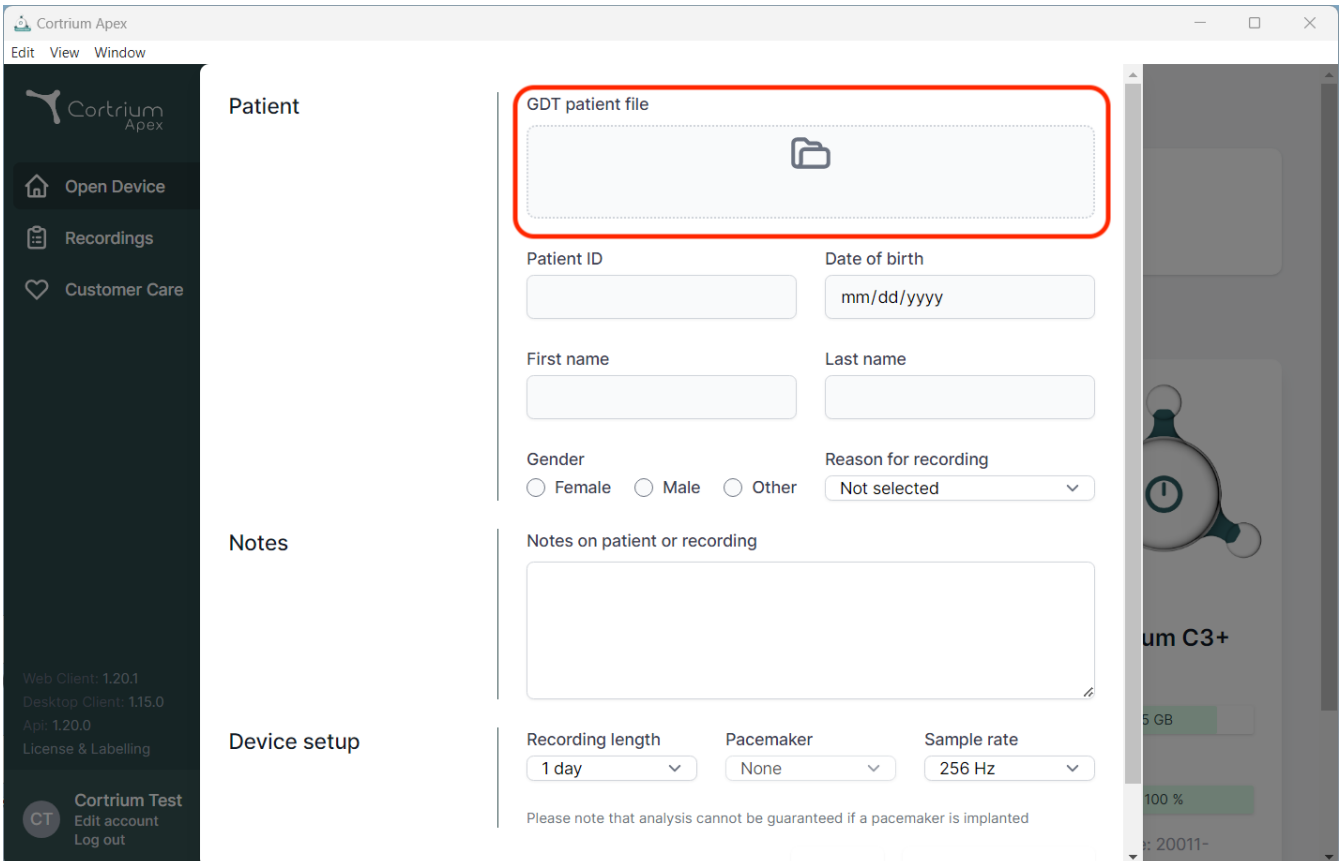
The screenshot displays the Cortrium Apex web application interface. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a form for entering patient information. The 'GDT patient file' field is highlighted with a red box, indicating where a file should be uploaded. Below this field are input fields for Patient ID, Date of birth (with a placeholder 'mm/dd/yyyy'), First name, and Last name. There are also radio buttons for Gender (Female, Male, Other) and a dropdown menu for Reason for recording (currently set to 'Not selected'). The Notes section has a text area for 'Notes on patient or recording'. The Device setup section includes dropdown menus for Recording length (set to '1 day'), Pacemaker (set to 'None'), and Sample rate (set to '256 Hz'). A disclaimer at the bottom of the Device setup section states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. At the bottom of the sidebar, there is a 'Cortrium Test' button and links for 'Edit account' and 'Log out'. The top of the interface shows the 'Cortrium Apex' logo and a menu with 'Edit', 'View', and 'Window' options.

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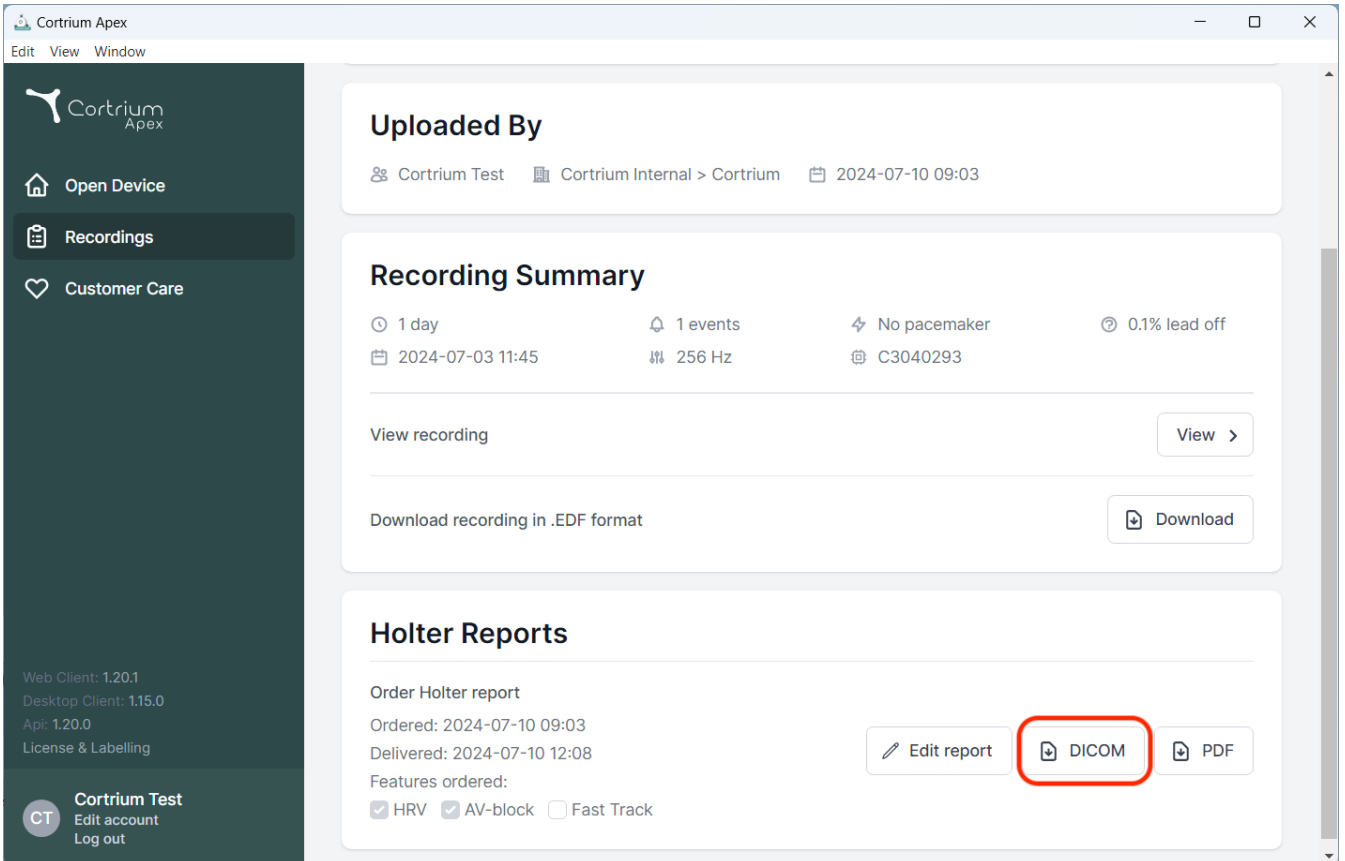
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' for 'Cortrium Internal > Cortrium' on '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' (2024-07-03 11:45), '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off' (C3040293). It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

Footer information includes: Web Client: 1.20.1, Desktop Client: 1.15.0, Api: 1.20.0, License & Labelling, and user 'CT Cortrium Test' with 'Edit account' and 'Log out' options.

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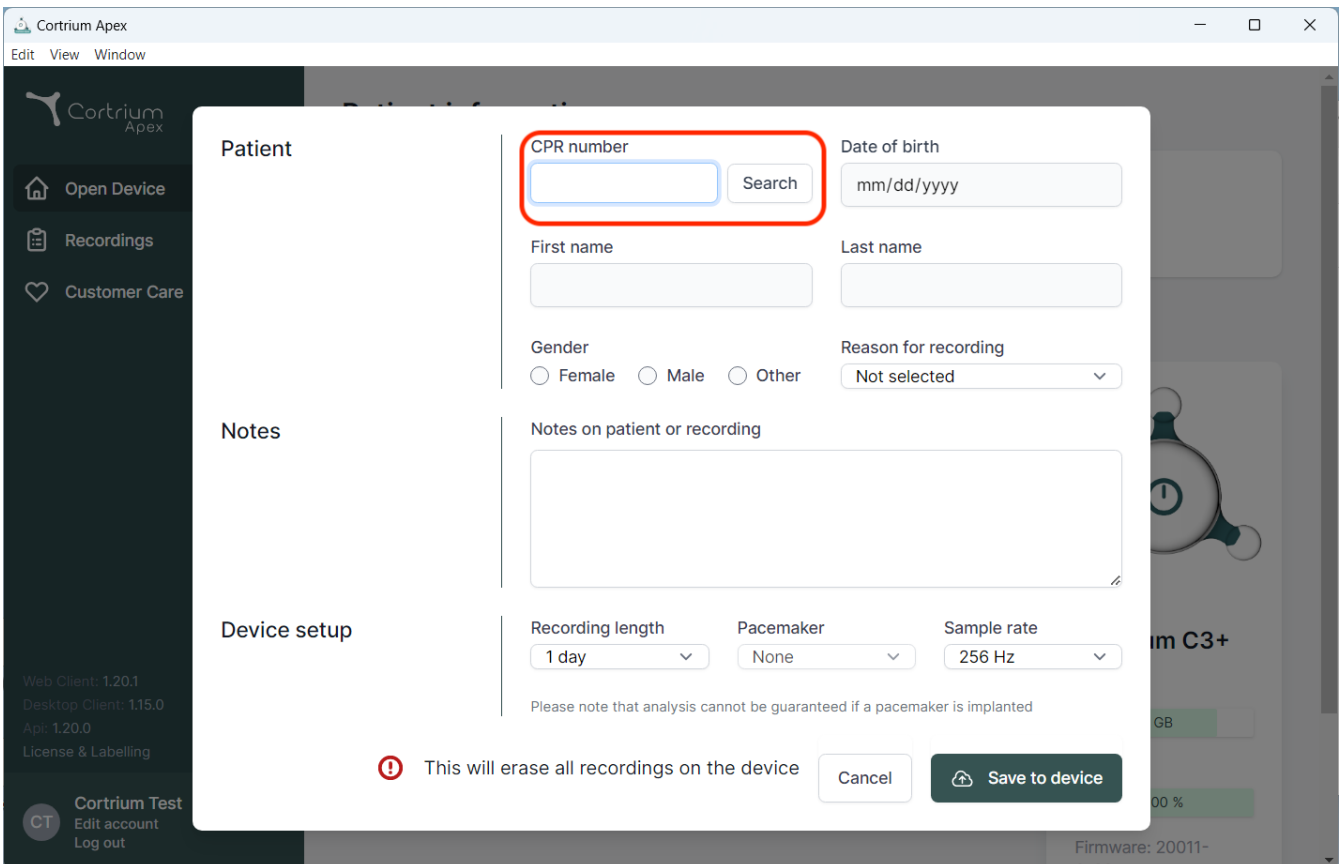The screenshot displays the Cortrium Apex web application interface. A modal window titled "Patient" is open, containing the following fields and options:

- CPR number:** A text input field with a "Search" button next to it. This field is highlighted with a red rectangular box.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently showing "Not selected".
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup section, there is a warning message: **!** This will erase all recordings on the device. At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

09/2024

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark
E-mail: info@cortrium.com Website: www.cortrium.com

Cortrium Apex UDI(01)05745000379033