Cortrium Apex 1.20

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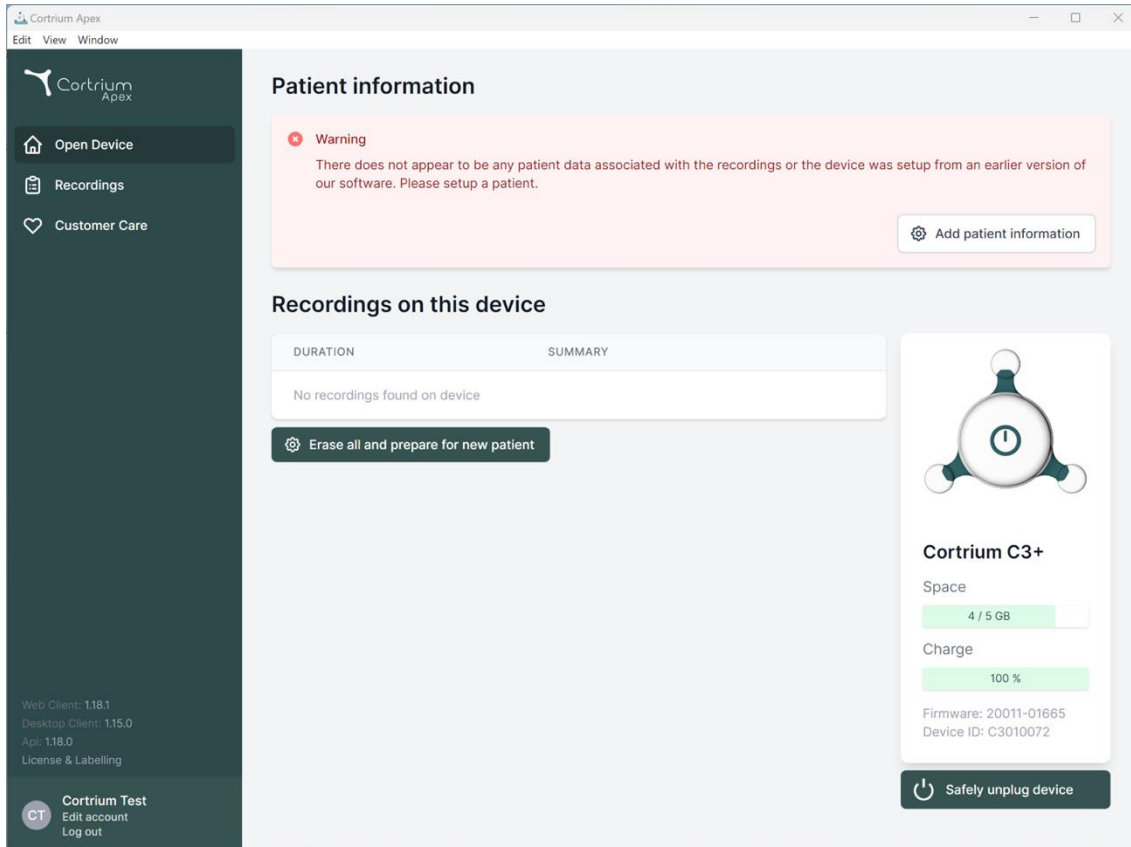
The screenshot shows the Cortrium Apex web interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. Below these are version numbers (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0) and a 'Cortrium Test' section with 'Edit account' and 'Log out' links. The main content area is titled 'Patient information' and features a warning message: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button. Below this is a 'Recordings on this device' section with a table header for 'DURATION' and 'SUMMARY', showing 'No recordings found on device' and an 'Erase all and prepare for new patient' button. On the right, a device status card for 'Cortrium C3+' displays 'Space: 4 / 5 GB' and 'Charge: 100 %', along with 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex
Edit View Window

Cortrium Apex

Open Device
Recordings
Customer Care

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: Female Male Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

⚠ This will erase all recordings on the device Cancel **Save to device**

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
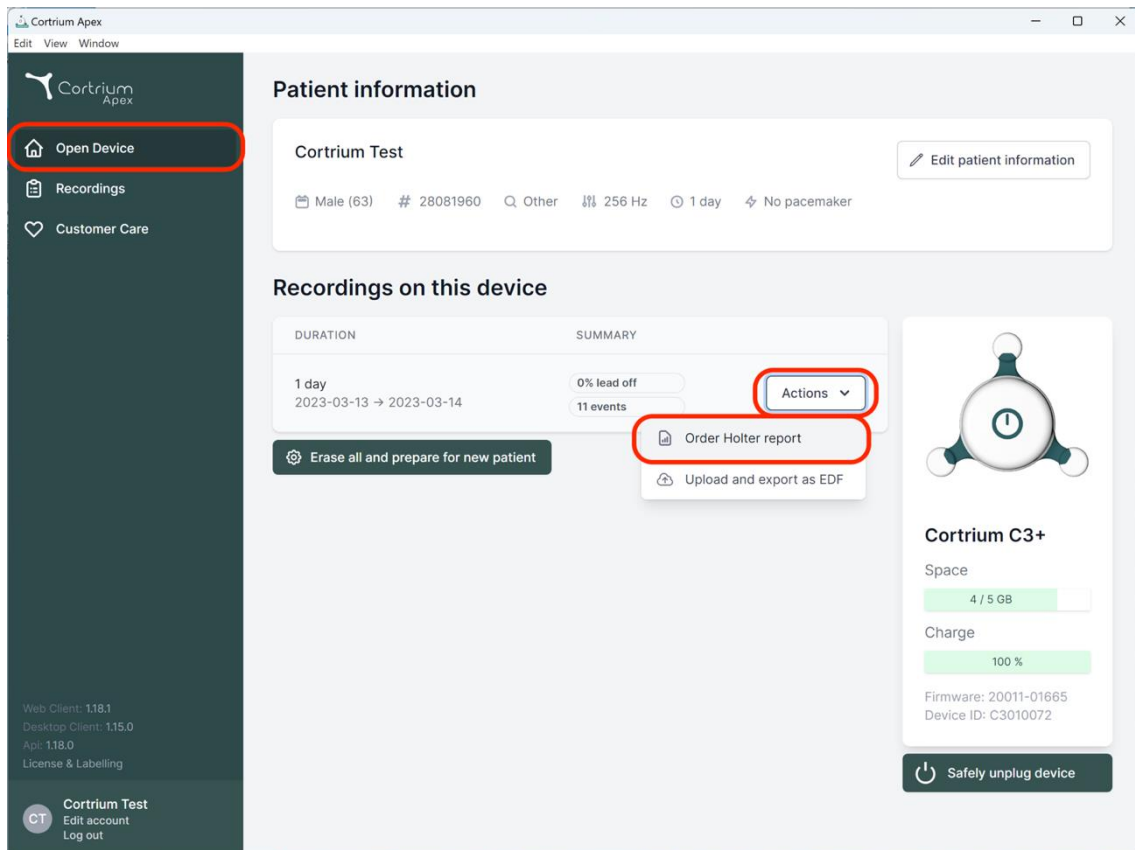
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device' (highlighted with a red box), 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male), age (63), ID (28081960), and recording duration (1 day). Below this is a table of 'Recordings on this device' with columns for 'DURATION' and 'SUMMARY'. The first recording is for '1 day' from '2023-03-13' to '2023-03-14', with '0% lead off' and '11 events'. An 'Actions' dropdown menu is open, showing 'Order Holter report' (highlighted with a red box) and 'Upload and export as EDF'. On the right, there is a device status card for 'Cortrium C3+' showing 'Space' (4 / 5 GB) and 'Charge' (100%). A 'Safely unplug device' button is at the bottom right. The bottom left of the sidebar shows the user's name 'Cortrium Test' and options to 'Edit account' and 'Log out'.

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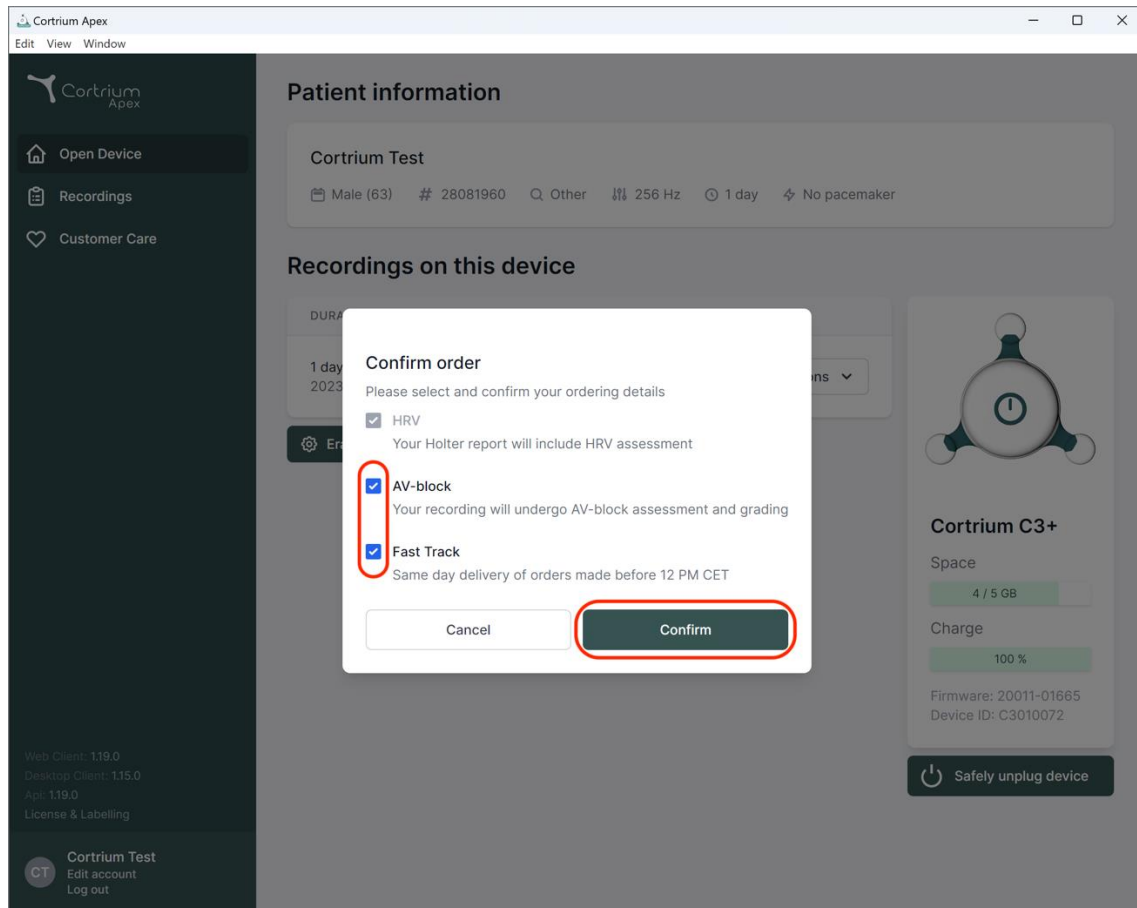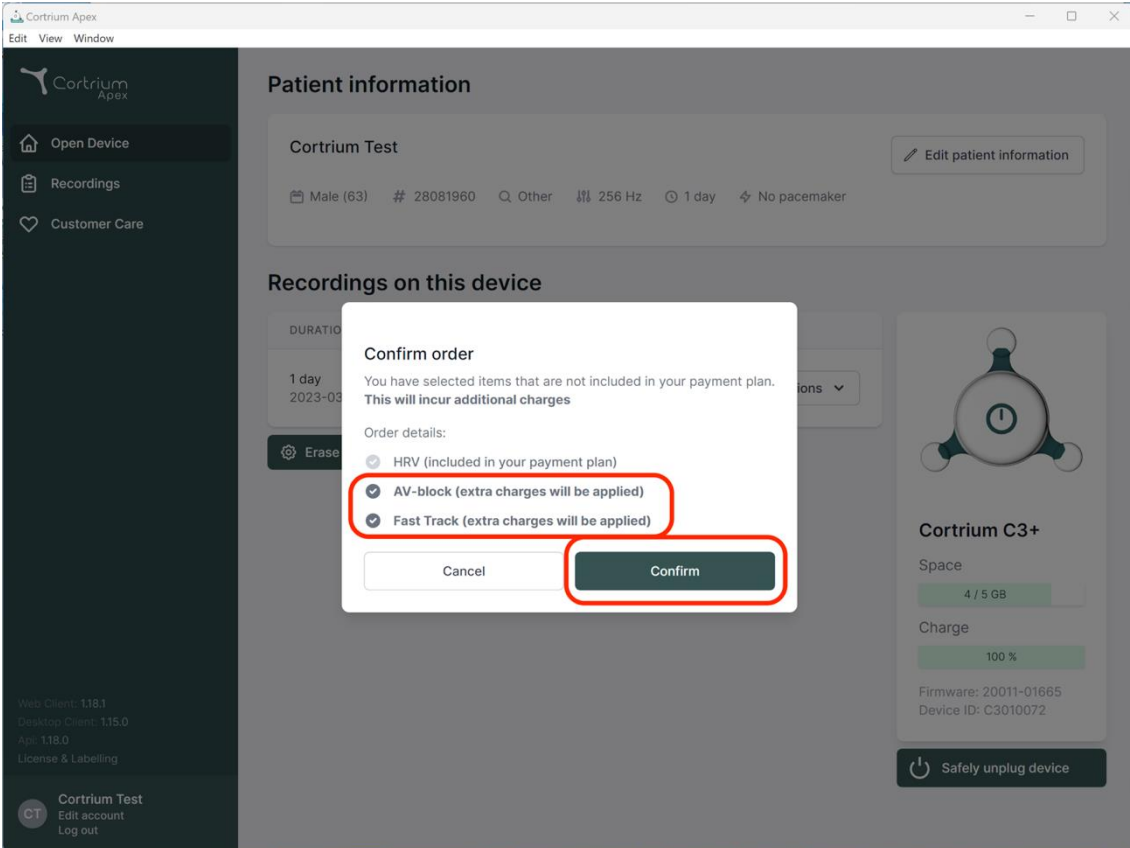
Erase

Confirm order

You have selected items that are not included in your payment plan.
This will incur additional charges

Order details:

- HRV (included in your payment plan)
- AV-block (extra charges will be applied)
- Fast Track (extra charges will be applied)

Cancel Confirm

Cortrium C3+

Space

4 / 5 GB

Charge

100 %

Firmware: 20011-01665


Device ID: C3010072

Safely unplug device

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
CT Edit account
Log out

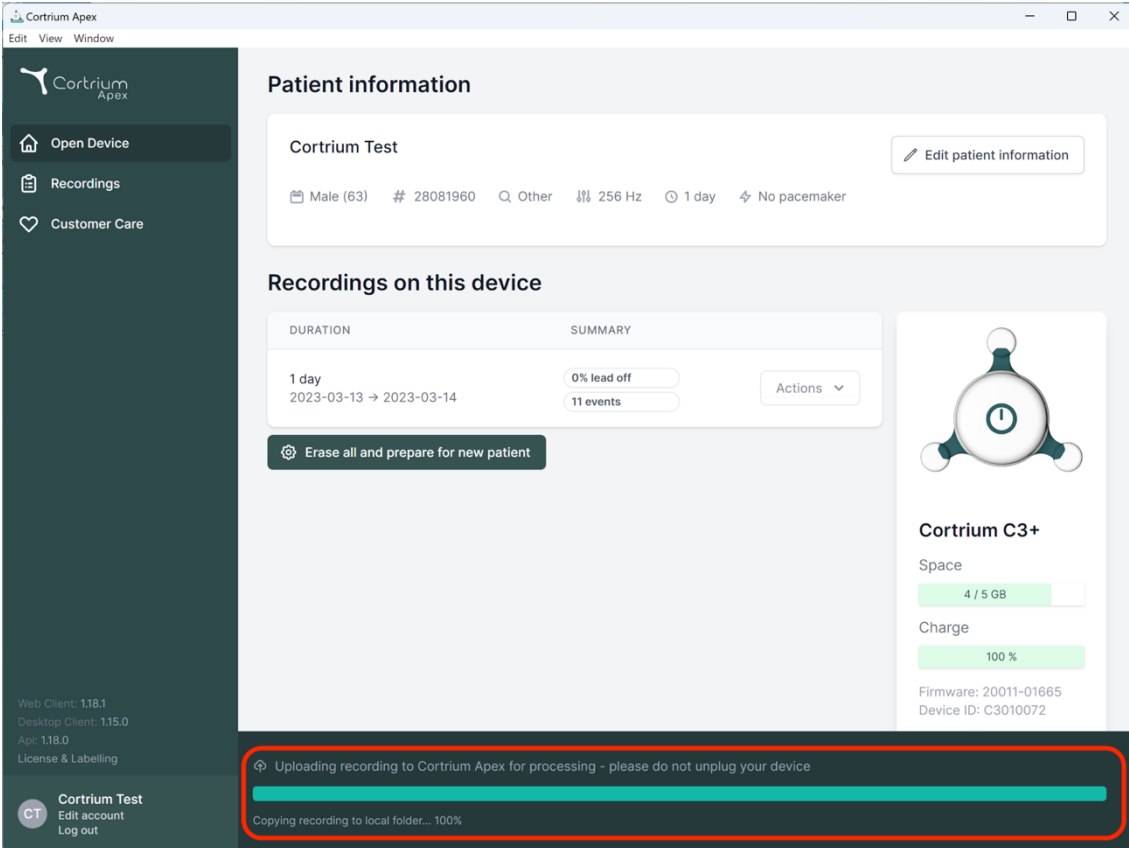Inspelningen laddas nu upp.

The screenshot shows the Cortrium Apex web application interface. The top navigation bar includes 'Edit', 'View', and 'Window'. The left sidebar contains 'Open Device', 'Recordings', and 'Customer Care'. The main content area is titled 'Patient information' and displays details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). A summary section shows '0% lead off' and '11 events'. A 'Recordings on this device' section includes a table with columns for 'DURATION' and 'SUMMARY'. A 'Cortrium C3+' device status panel shows 'Space: 4 / 5 GB' and 'Charge: 100%'. A red box highlights a progress bar at the bottom of the interface, indicating 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'.

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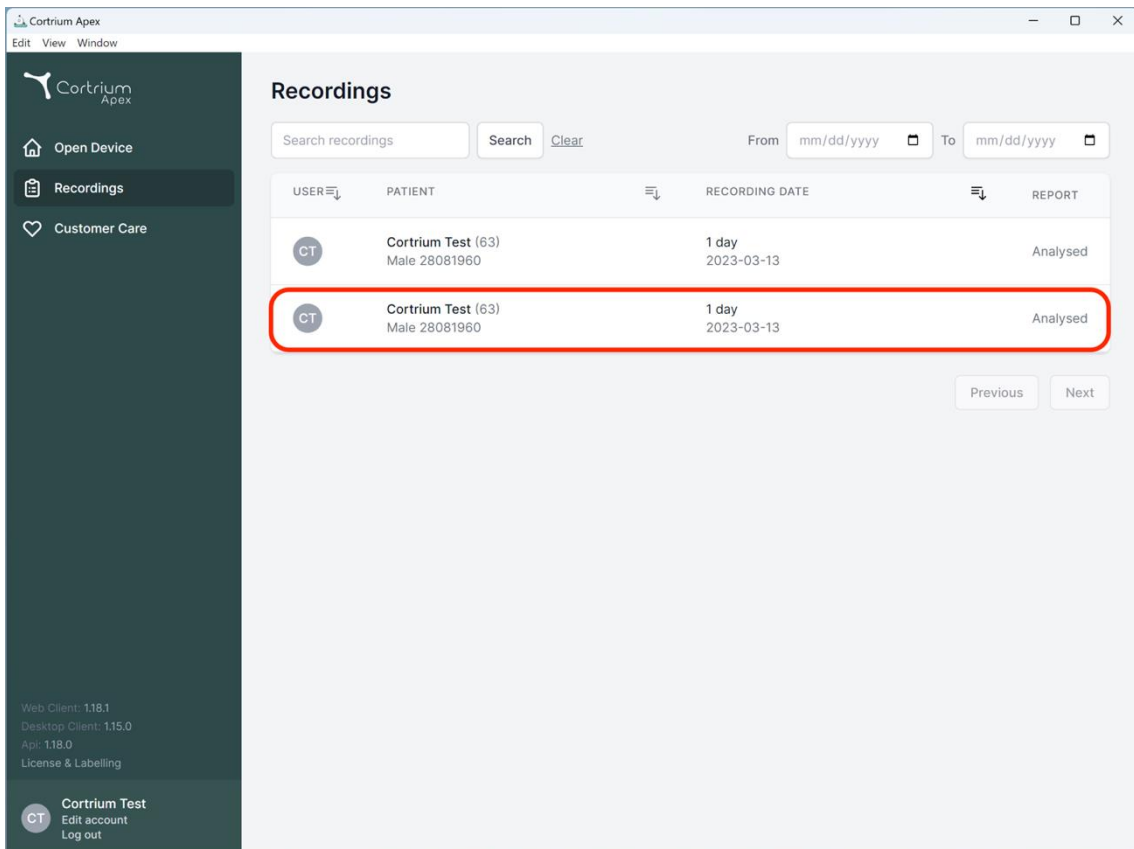
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings (selected), and Customer Care. At the bottom of the sidebar, there is a user profile for 'Cortrium Test' with options to edit the account or log out. The main content area is titled 'Recordings' and features a search bar with a 'Search' button and a 'Clear' link. Below the search bar is a table with columns for USER, PATIENT, RECORDING DATE, and REPORT. The table contains two rows of data, both for 'Cortrium Test (63) Male 28081960' with a '1 day' recording date of '2023-03-13' and a status of 'Analysed'. The second row is highlighted with a red rectangular border. At the bottom right of the table area, there are 'Previous' and 'Next' navigation buttons.

| USER | PATIENT | RECORDING DATE | REPORT |
|------|-------------------------------------|---------------------|----------|
| CT | Cortrium Test (63) Male 28081960 | 1 day 2023-03-13 | Analysed |
| CT | Cortrium Test (63) Male 28081960 | 1 day 2023-03-13 | Analysed |

Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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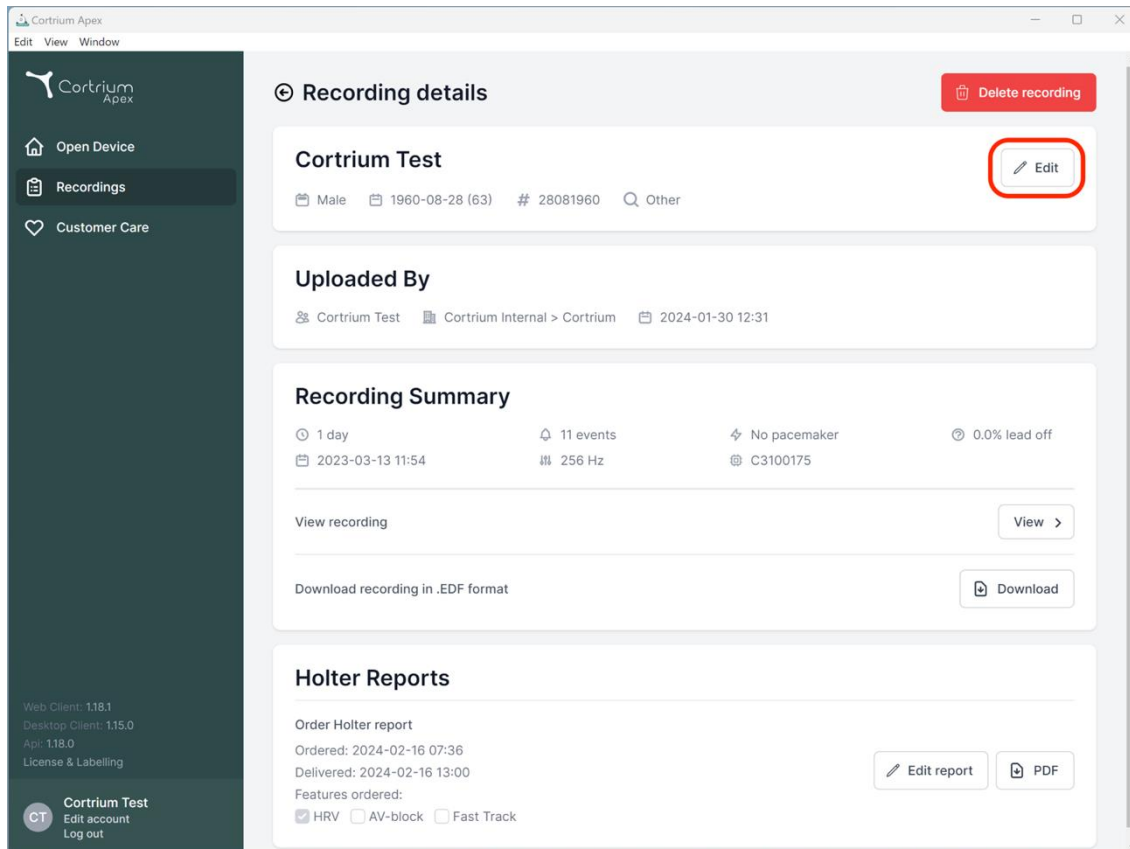
Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary
1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

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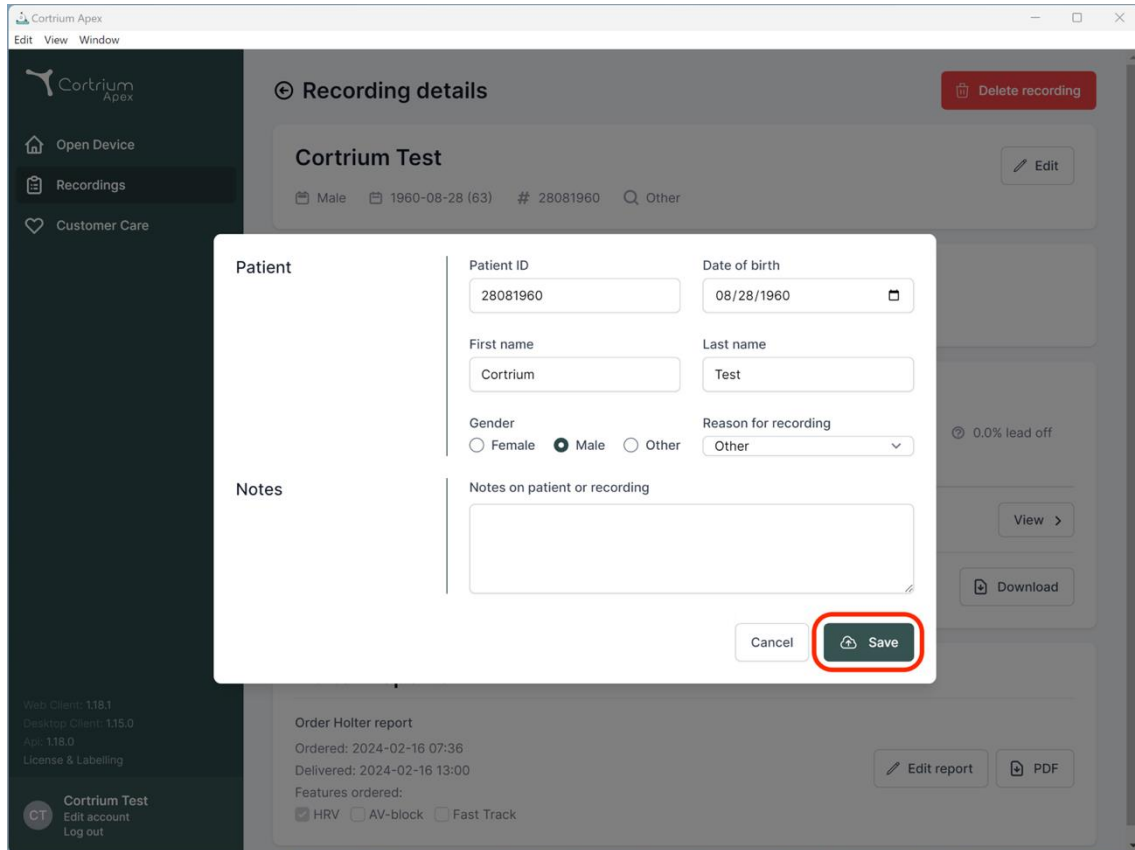
Holter Reports

Order Holter report
Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00
Features ordered:
 HRV AV-block Fast Track Edit report PDF

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
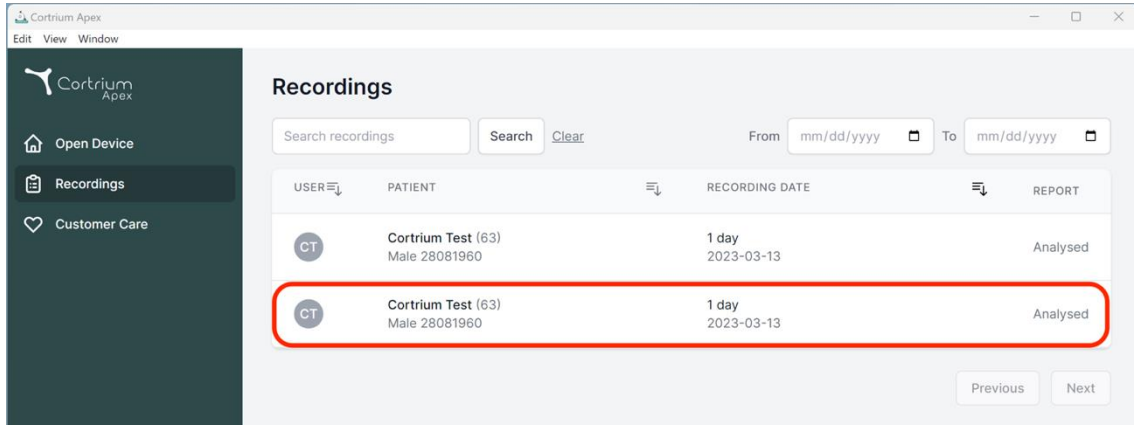
The screenshot shows the 'Recordings' section of the Cortrium Apex interface. A table lists recordings with columns for USER, PATIENT, RECORDING DATE, and REPORT. The second row is highlighted with a red border.

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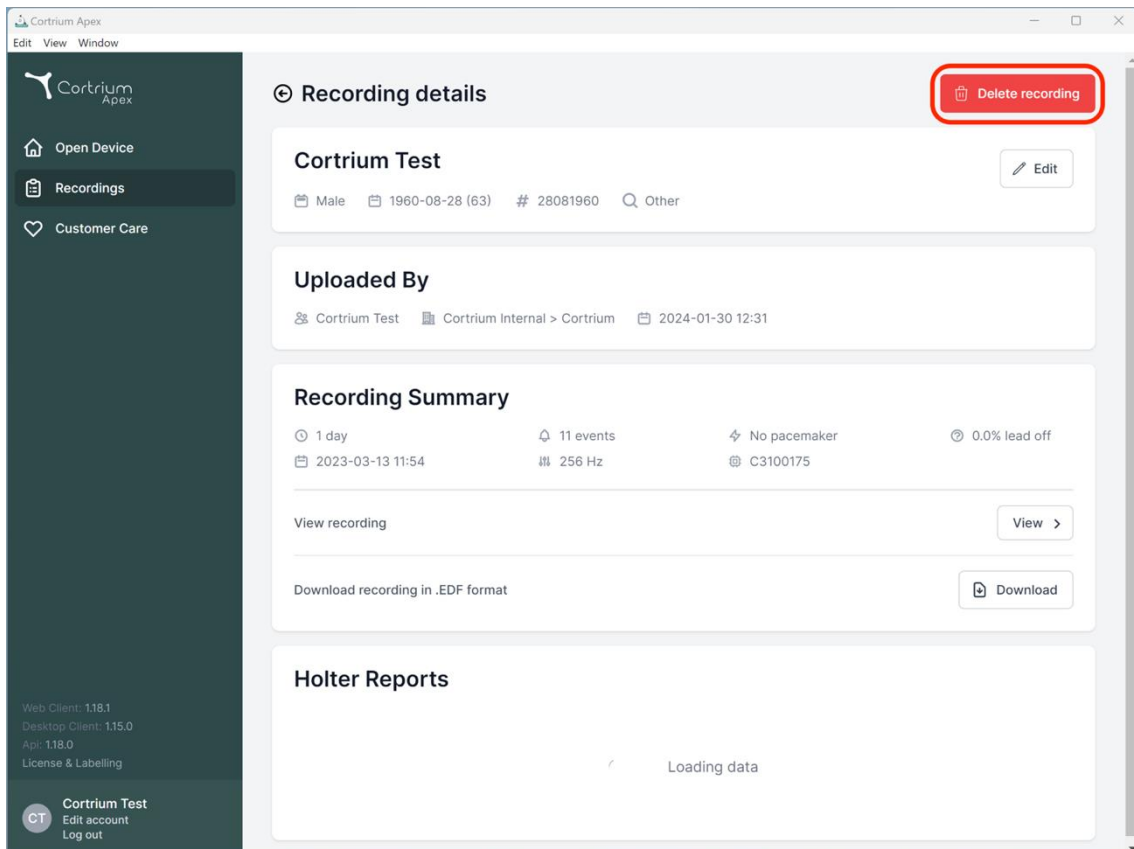The screenshot shows the 'Recording details' page for a specific recording. A red box highlights the 'Delete recording' button in the top right corner.

Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

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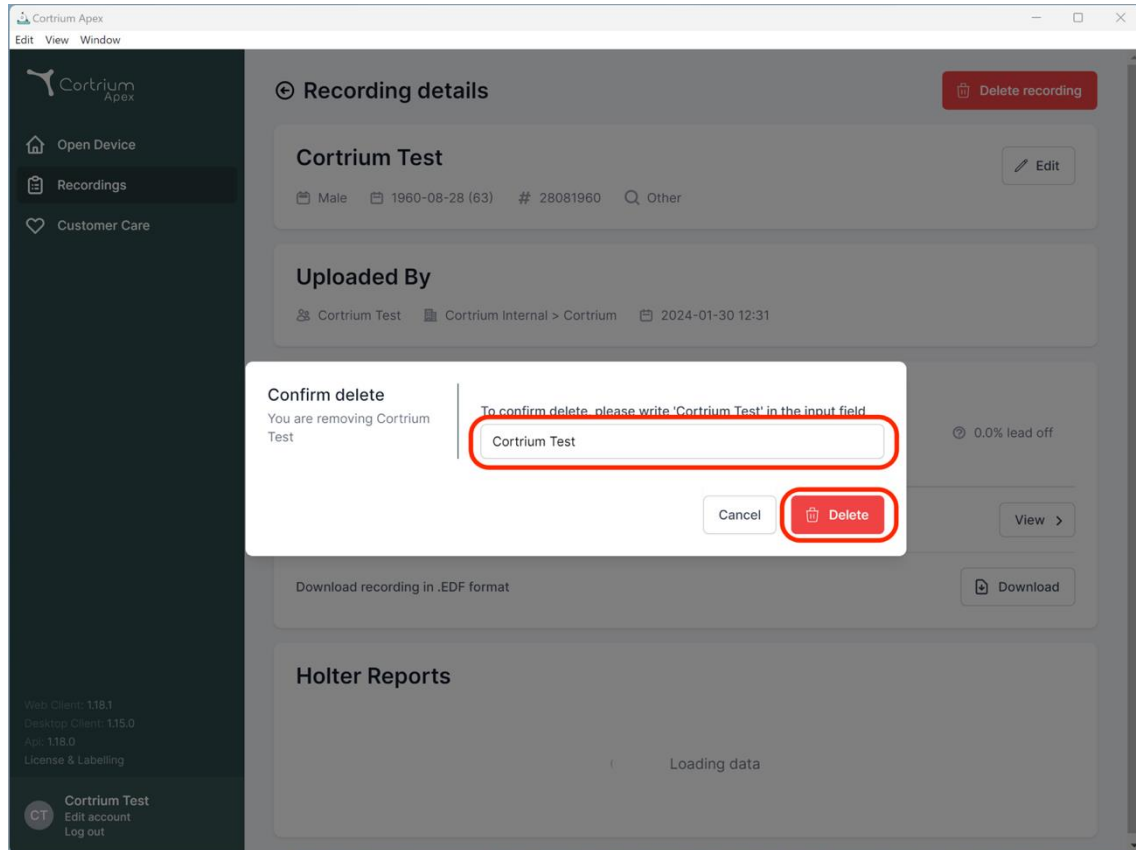
Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Confirm delete
You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

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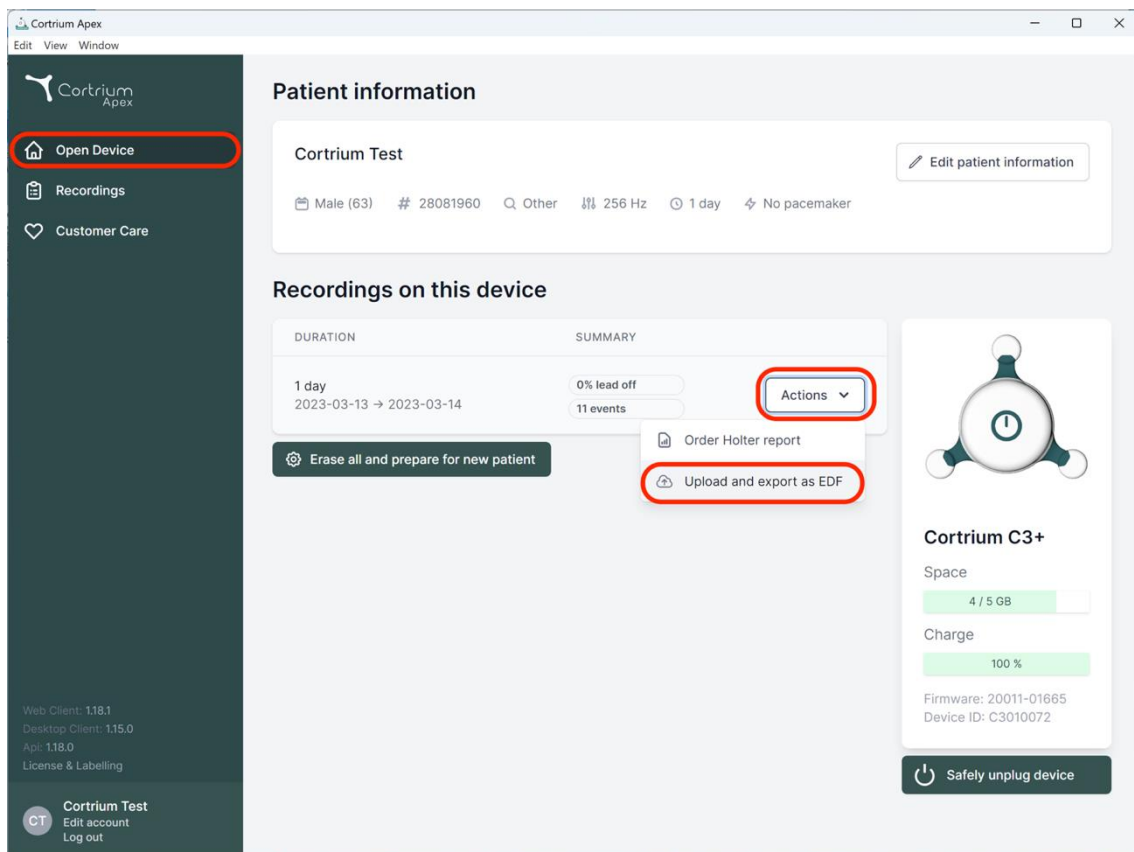
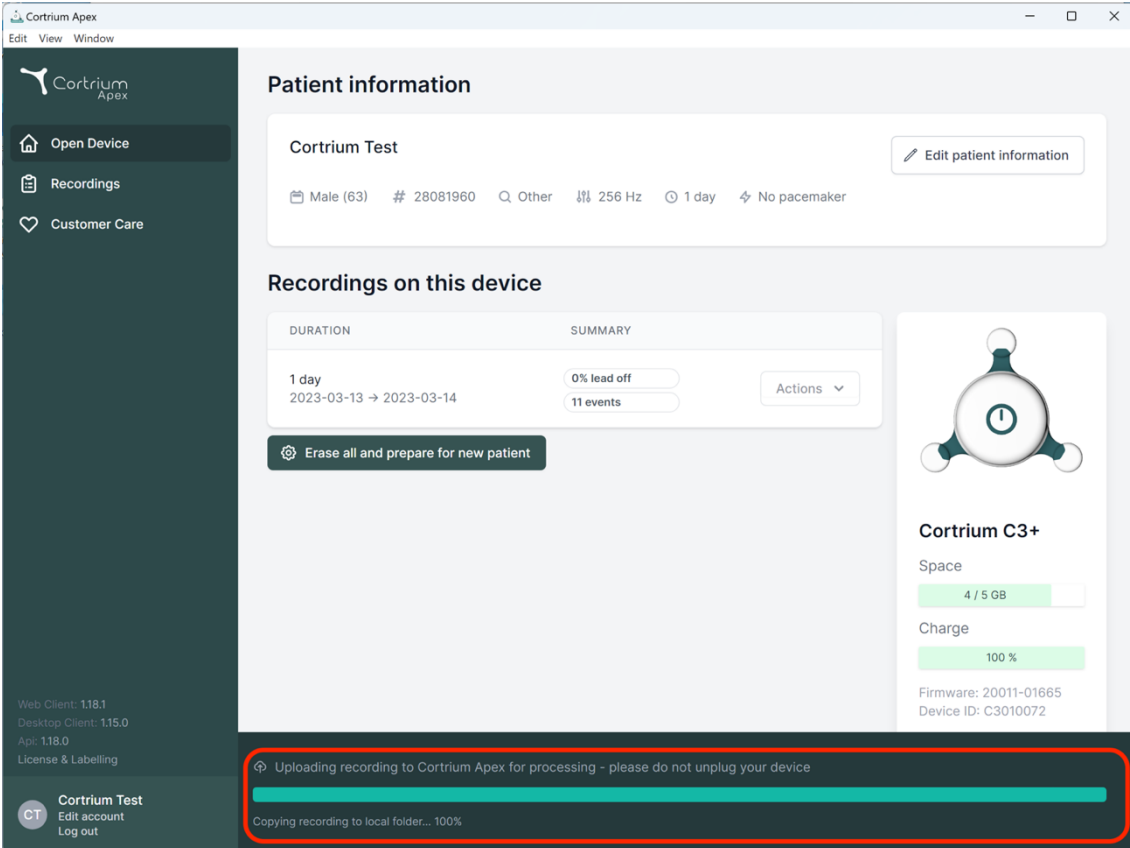
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

| DURATION | SUMMARY |
|----------------------------------|--|
| 1 day 2023-03-13 → 2023-03-14 | 0% lead off 11 events Actions |

[Erase all and prepare for new patient](#)

Uploading recording to Cortrium Apex for processing - please do not unplug your device

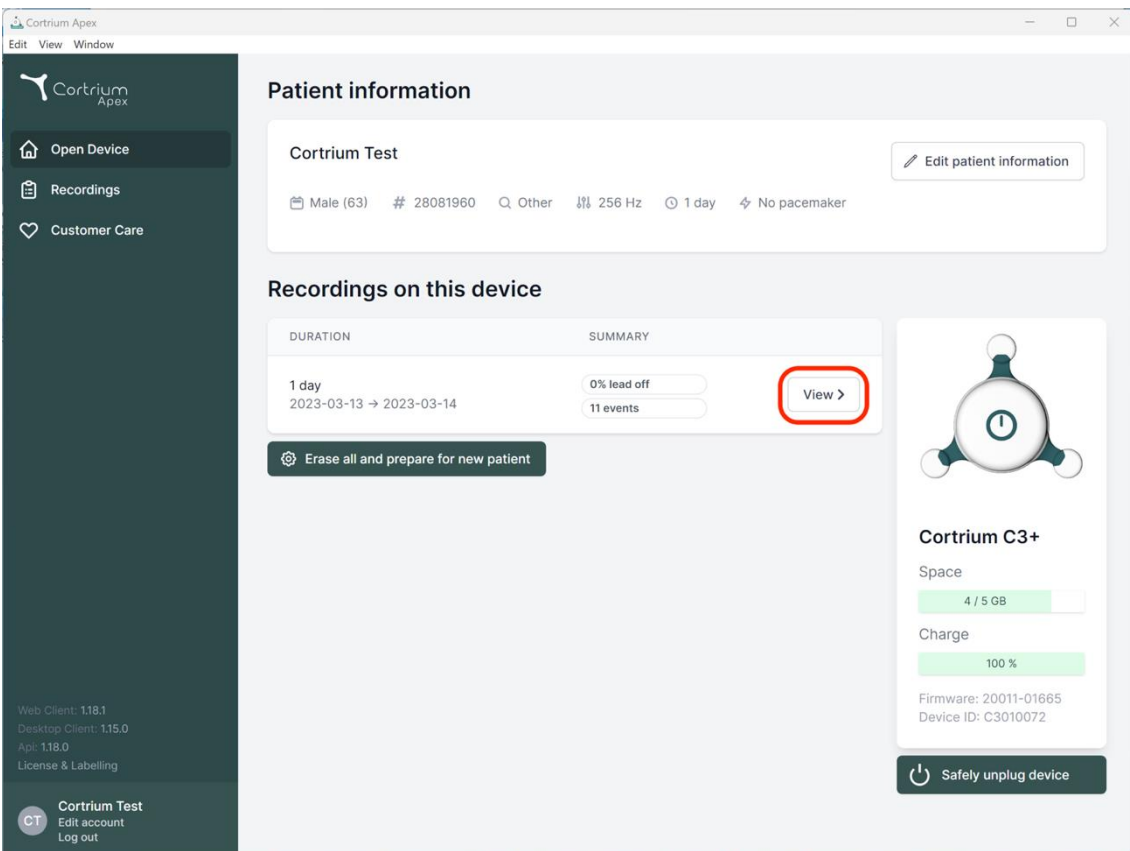
Copying recording to local folder... 100%

Cortrium C3+

Space: 4 / 5 GB

Charge: 100 %

Firmware: 20011-01665
Device ID: C3010072

Cortrium Apex

Edit View Window

Cortrium Apex

Open Device

Recordings

Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

Cortrium Test
Edit account
Log out

Patient information

Cortrium Test [Edit patient information](#)

Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

| DURATION | SUMMARY |
|----------------------------------|--|
| 1 day 2023-03-13 → 2023-03-14 | 0% lead off 11 events View > |

[Erase all and prepare for new patient](#)

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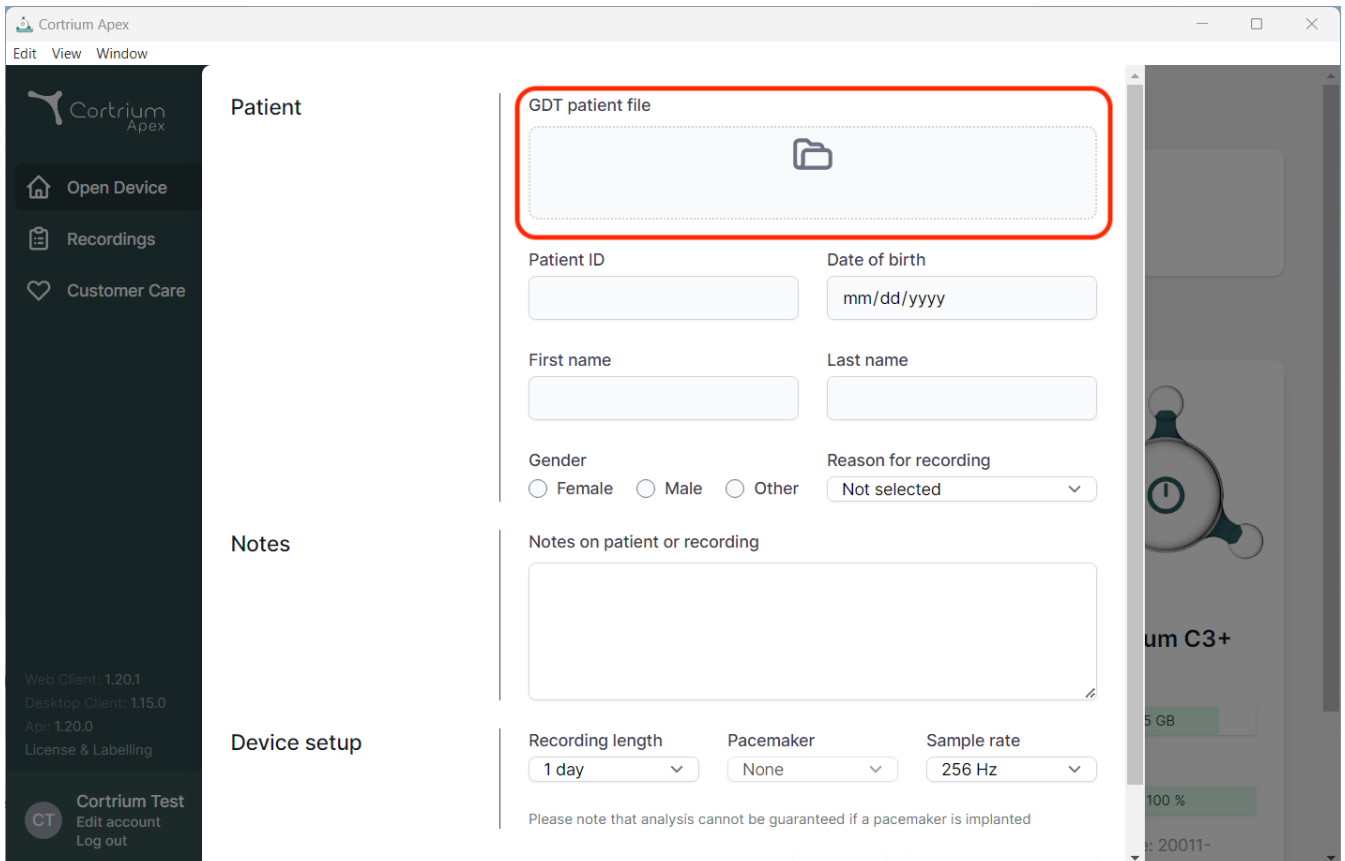
The screenshot shows the Cortrium Apex web application interface. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a form for entering patient information. A red box highlights the 'GDT patient file' field, which is a file upload area. Below this are fields for Patient ID, Date of birth (mm/dd/yyyy), First name, Last name, Gender (Female, Male, Other), and Reason for recording (Not selected). The Notes section has a text area for 'Notes on patient or recording'. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The bottom left corner shows the user's name 'Cortrium Test' and options to Edit account and Log out. The top left corner shows the application title 'Cortrium Apex' and menu options 'Edit', 'View', and 'Window'.

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Web Client: 1.20.1
Desktop Client: 1.15.0
App: 1.20.0
License & Labelling

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
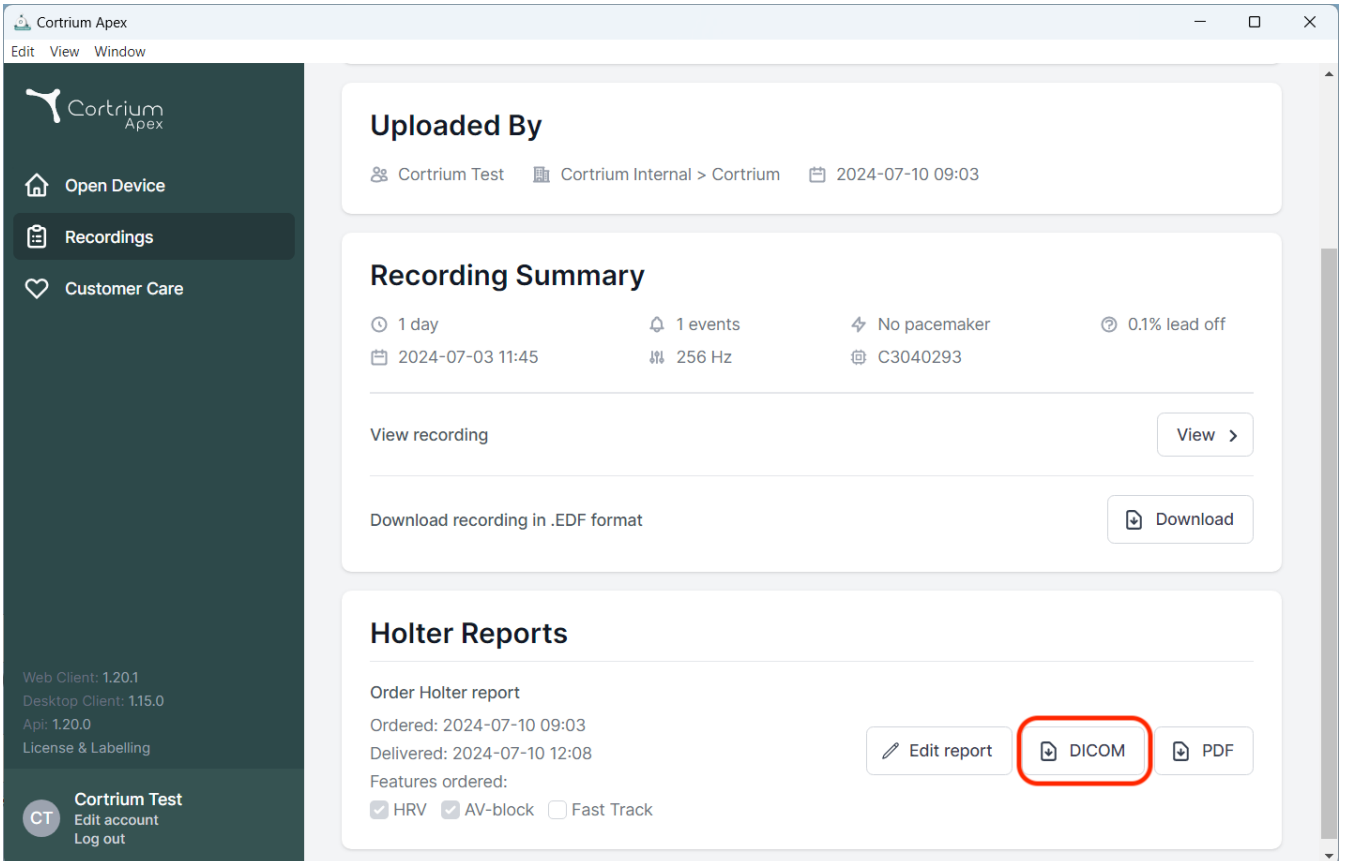
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' for 'Cortrium Internal > Cortrium' on '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' rate, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red box), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. Version information for Web Client (1.20.1), Desktop Client (1.15.0), and Api (1.20.0) is also visible.

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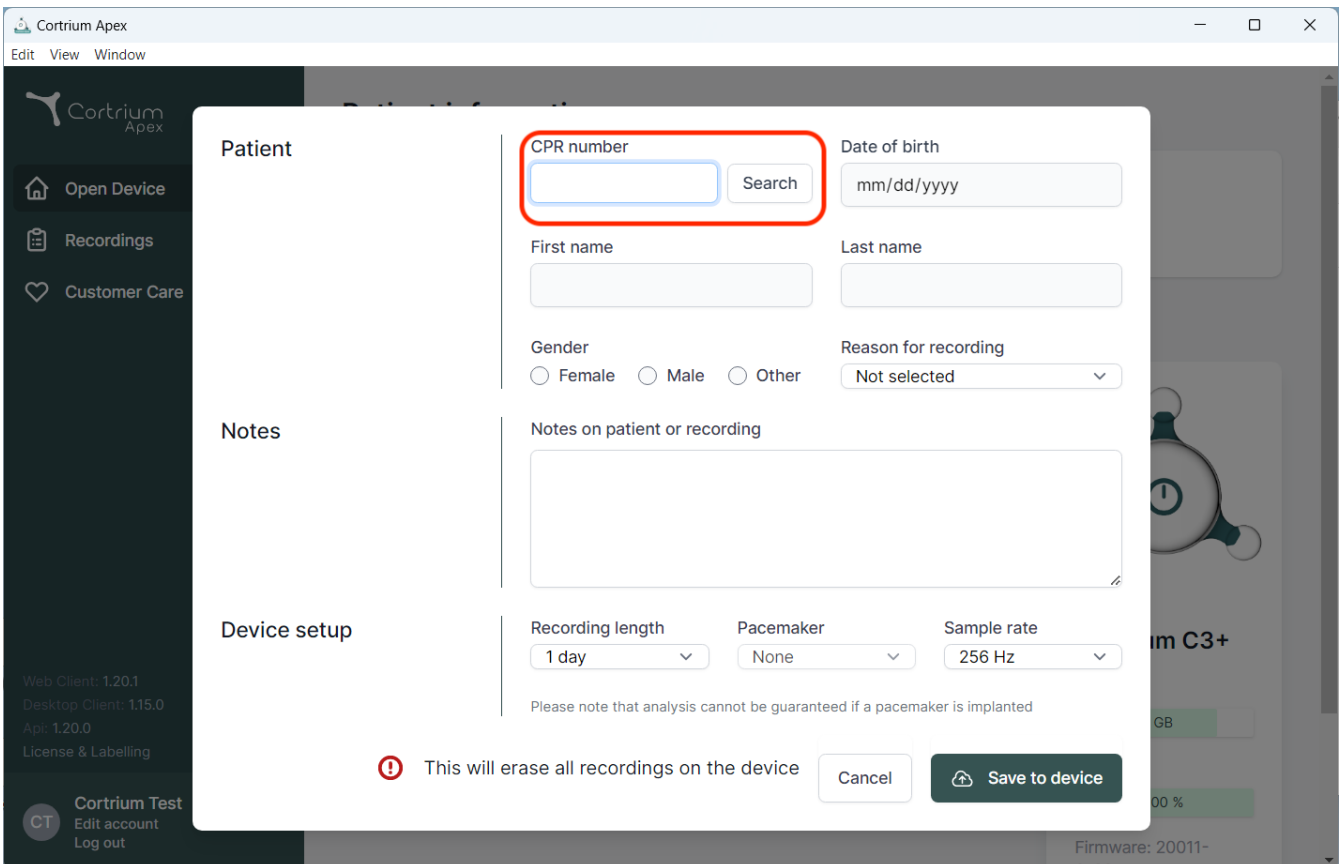The screenshot shows the Cortrium Apex software interface. A modal window titled "Patient" is open, containing the following fields and options:

- CPR number:** A text input field with a "Search" button next to it. This field is highlighted with a red box.
- Date of birth:** A text input field with the placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu currently set to "Not selected".
- Notes:** A large text area labeled "Notes on patient or recording".
- Device setup:** Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the device setup section, there is a warning message: "⚠ This will erase all recordings on the device". At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

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