Cortrium Apex 1.20

Bruksanvisning

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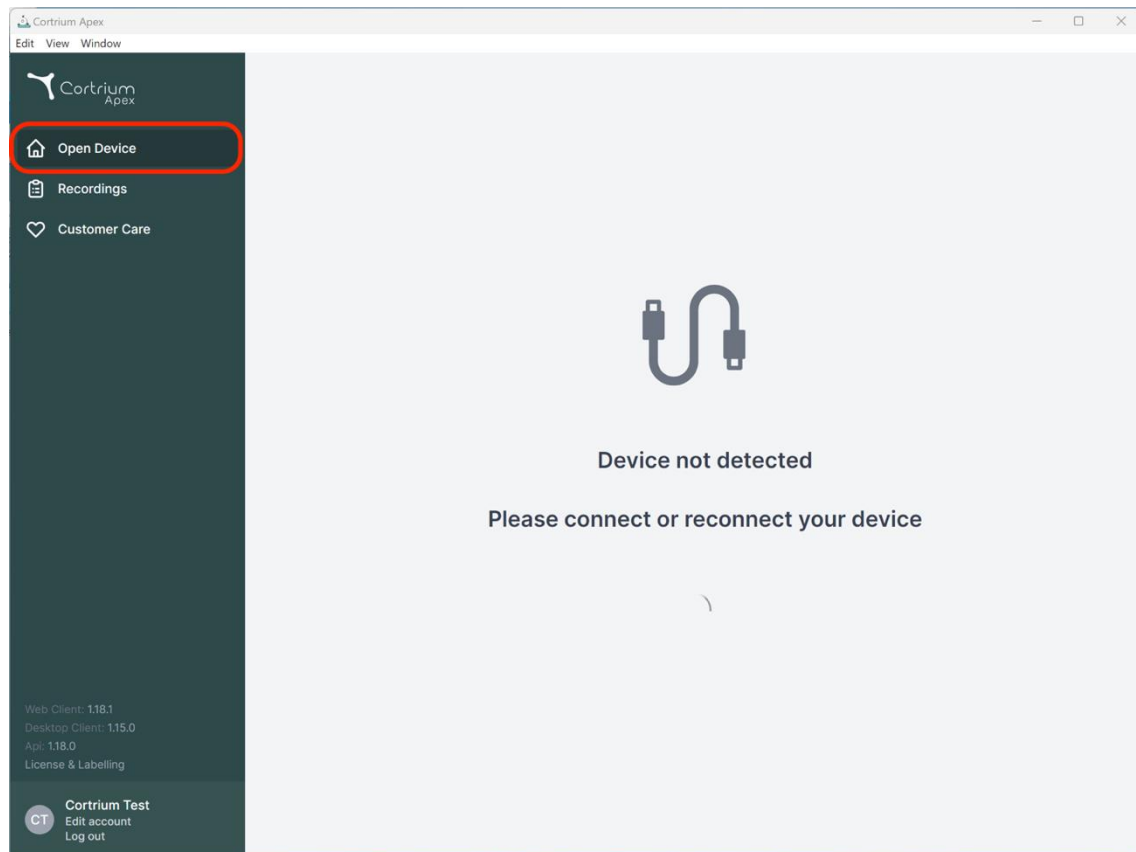
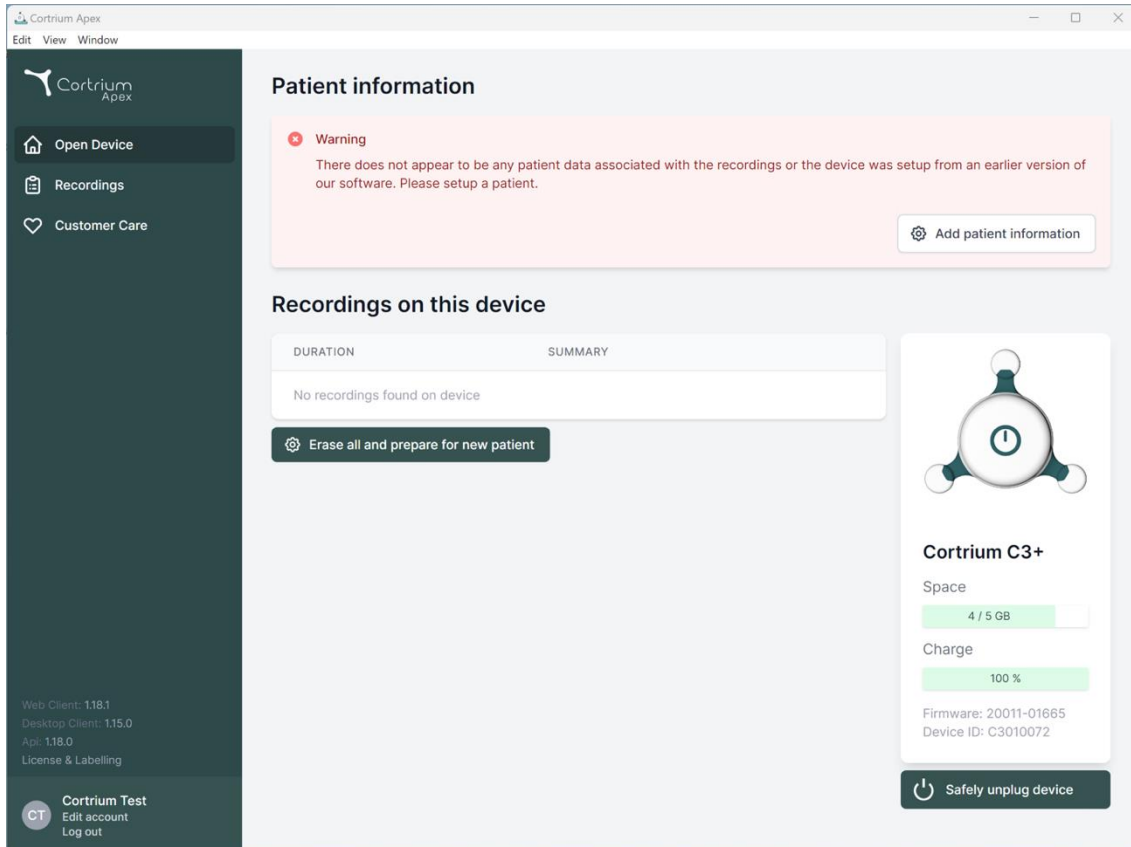
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. Below these are version numbers for Web Client (1.18.1), Desktop Client (1.15.0), and App (1.18.0), along with a 'License & Labelling' link. At the bottom of the sidebar is a 'Cortrium Test' section with 'Edit account' and 'Log out' options.

The main content area is titled 'Patient information' and features a red warning box: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' An 'Add patient information' button is located to the right of the warning.

Below the warning is a section titled 'Recordings on this device' with a table with columns 'DURATION' and 'SUMMARY'. The table is empty, displaying 'No recordings found on device'. A button 'Erase all and prepare for new patient' is positioned below the table.

On the right side of the main area is a device status card for 'Cortrium C3+'. It shows a battery icon, 'Space' usage at '4 / 5 GB', 'Charge' at '100%', and 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex
Edit View Window

Cortrium Apex

Open Device
Recordings
Customer Care

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: Female Male Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

⚠ This will erase all recordings on the device

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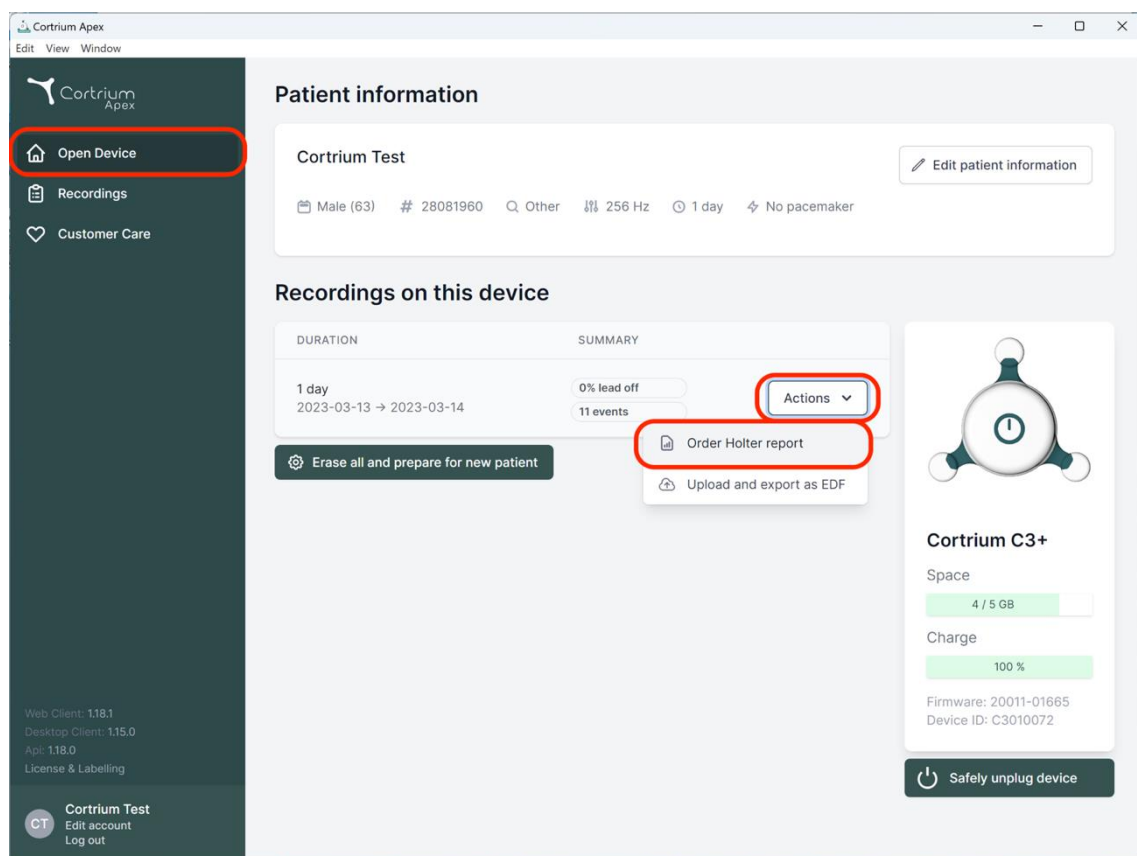
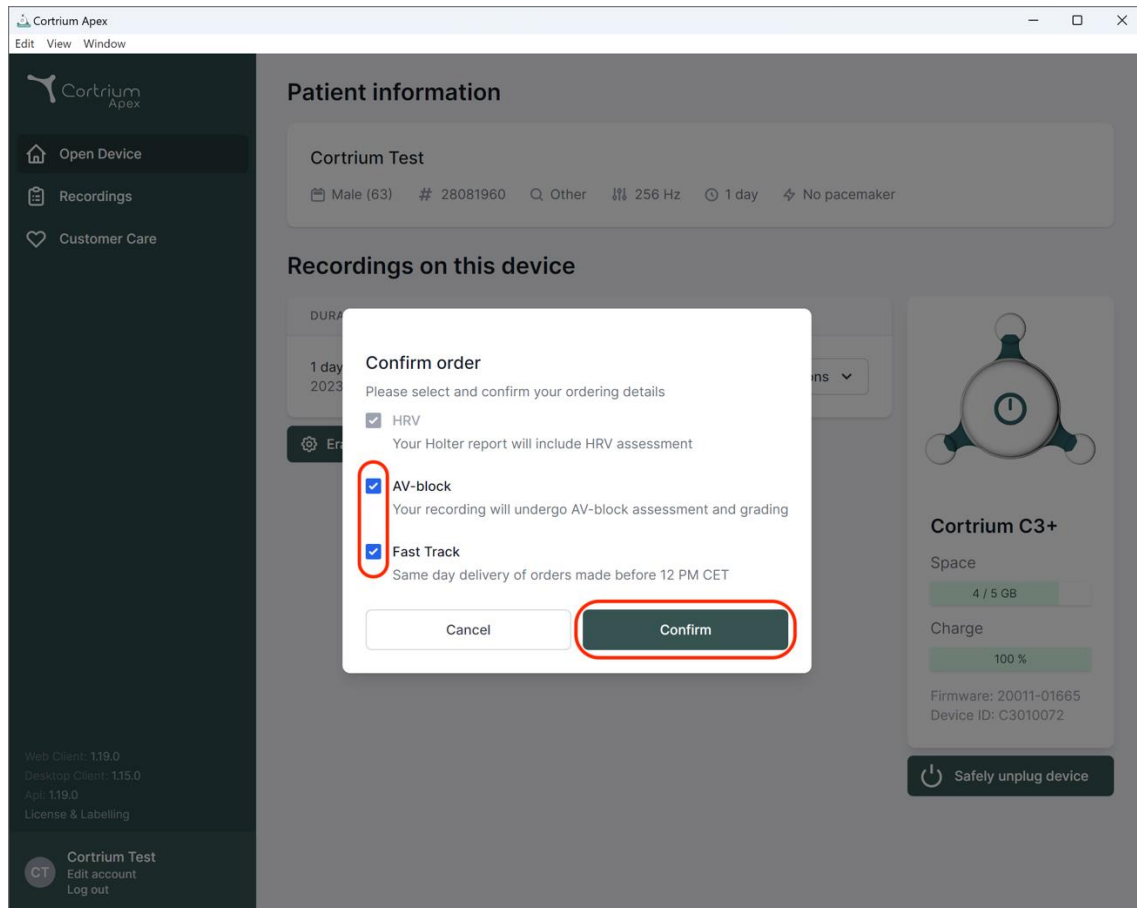
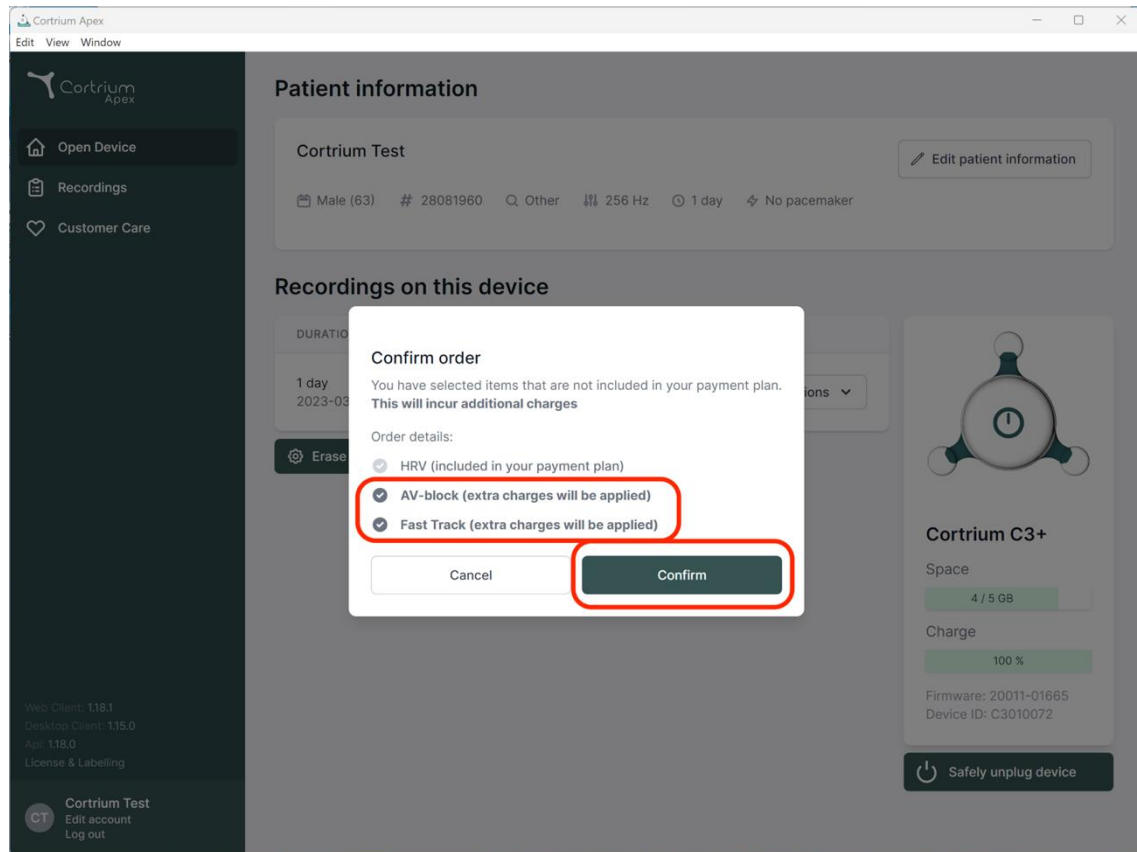
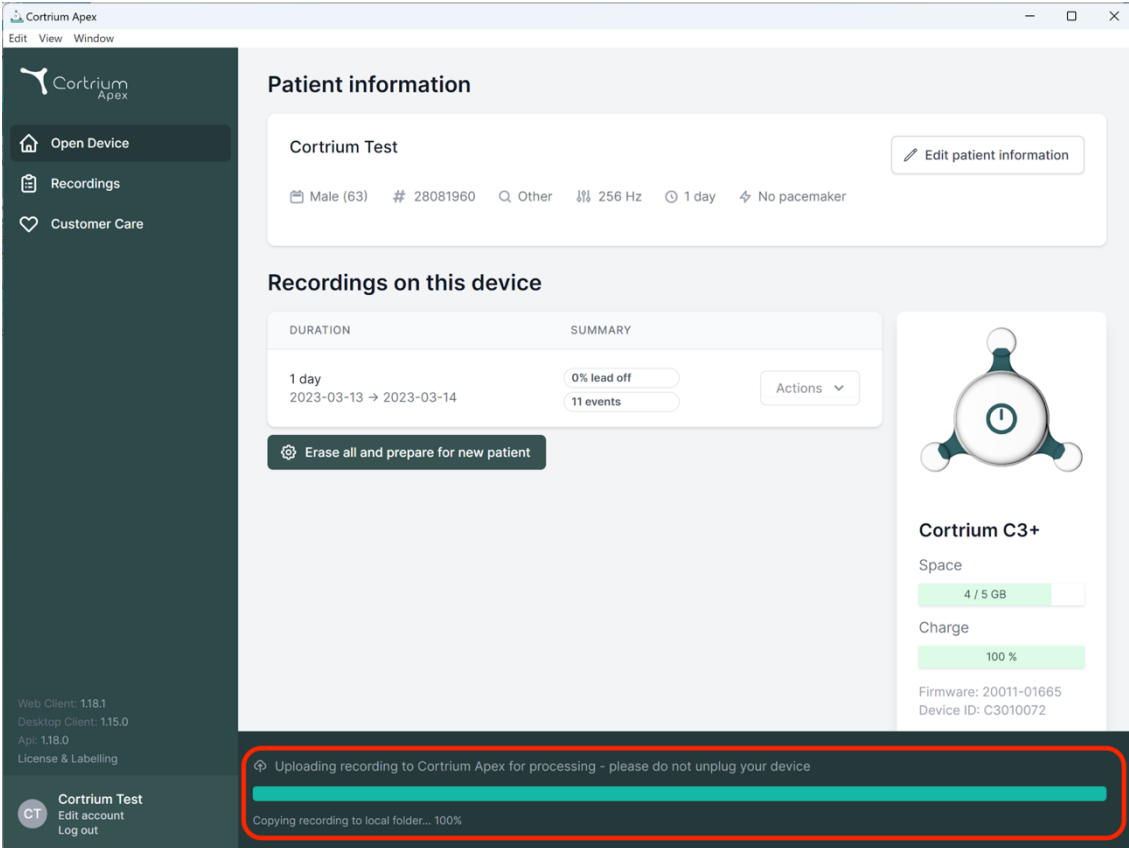
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and device settings (256 Hz, 1 day, No pacemaker). Below this is a 'Recordings on this device' section with a table showing a 1-day recording from 2023-03-13 to 2023-03-14, with 0% lead off and 11 events. A 'Copy' button is visible. To the right, a 'Cortrium C3+' device status card shows 4/5 GB space and 100% charge. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'. The sidebar also contains version information and user options like 'Edit account' and 'Log out'.

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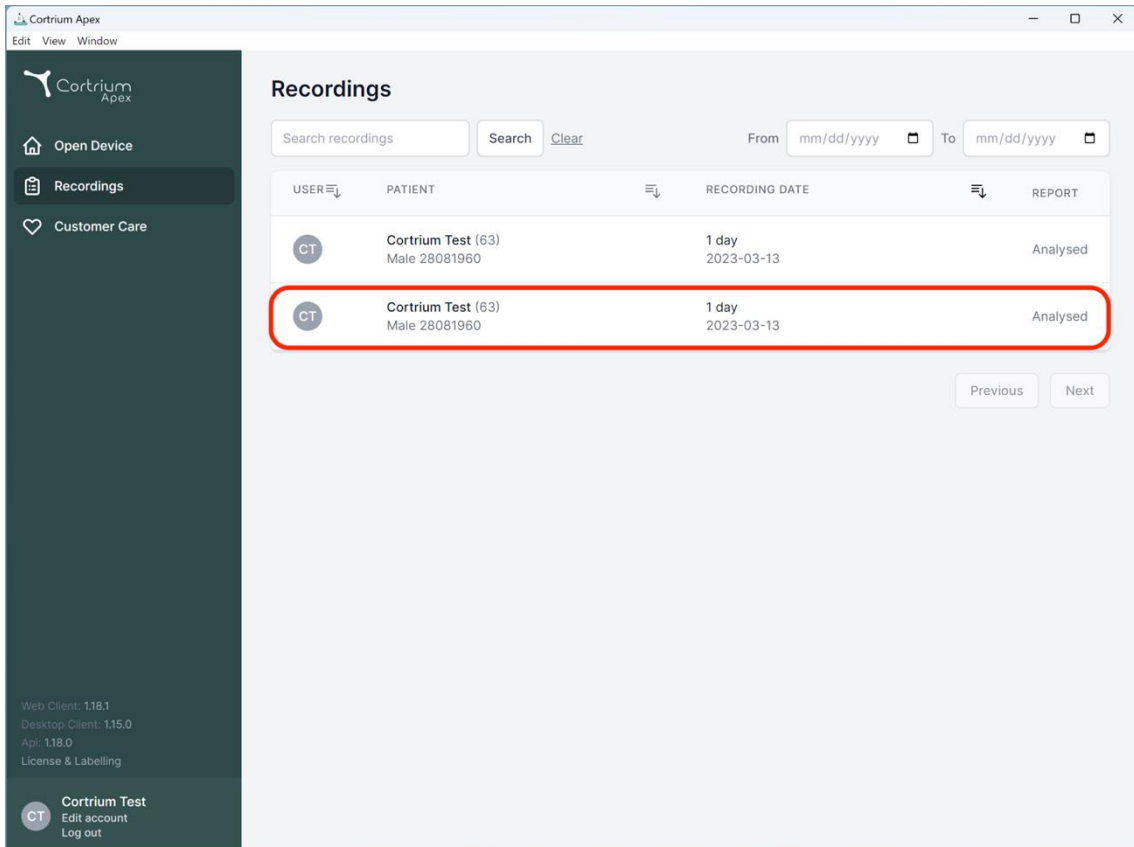
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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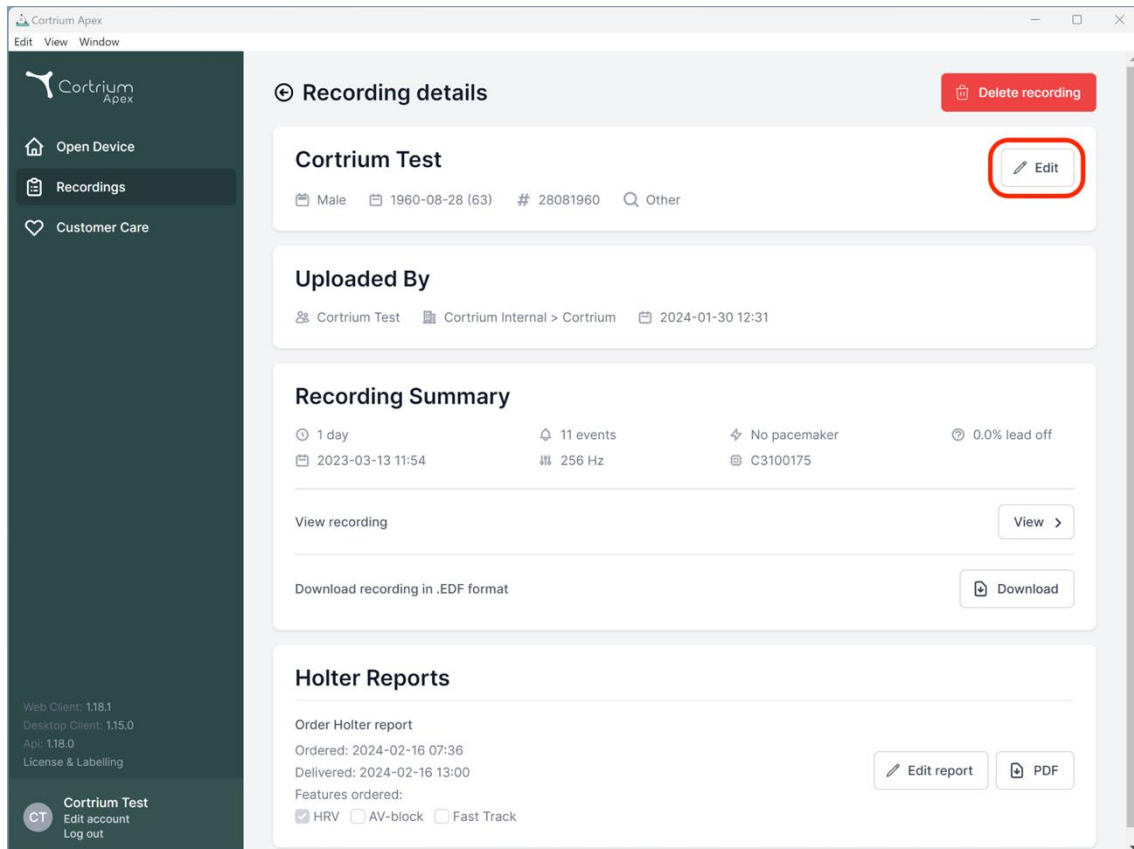
Du kan redigere pasientinformasjonen ved å klikke på knappen «Edit» (Rediger).

Recording details Delete recording

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By
Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

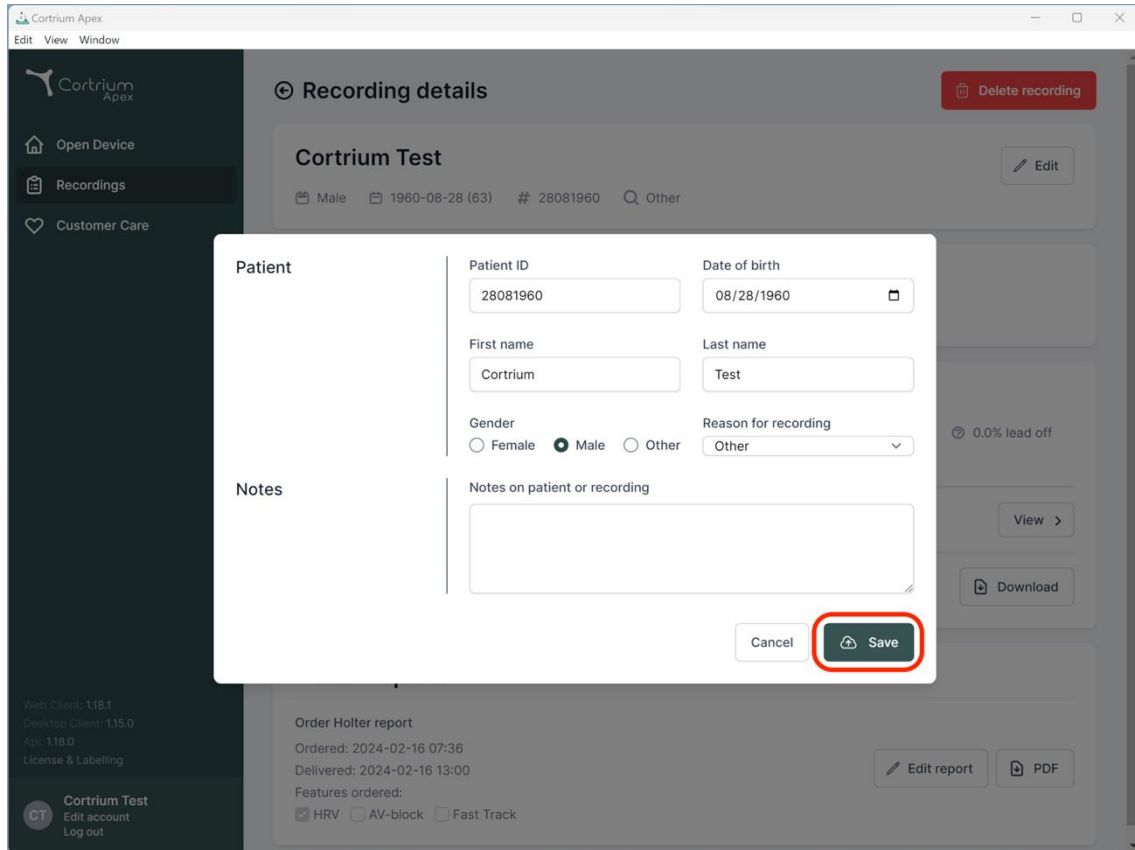
Recording Summary
1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

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Holter Reports
Order Holter report
Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00
Features ordered:
 HRV AV-block Fast Track Edit report PDF

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Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: Female Male Other Reason for recording: Other

Notes

Notes on patient or recording

Cancel Save

Web Client: 118.1 Desktop Client: 115.0 App: 118.0 License & Labelling

CT Cortrium Test Edit account Log out

Order Holter report

Ordered: 2024-02-16 07:36 Delivered: 2024-02-16 13:00

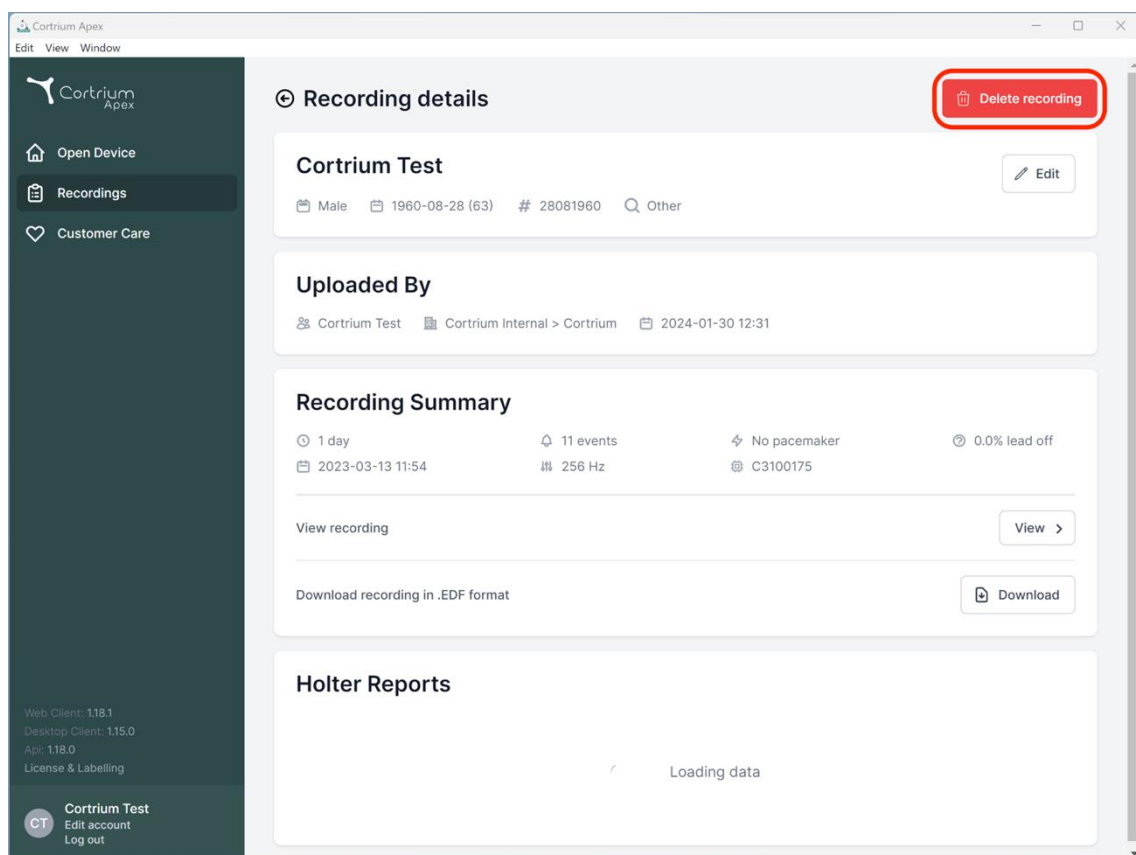
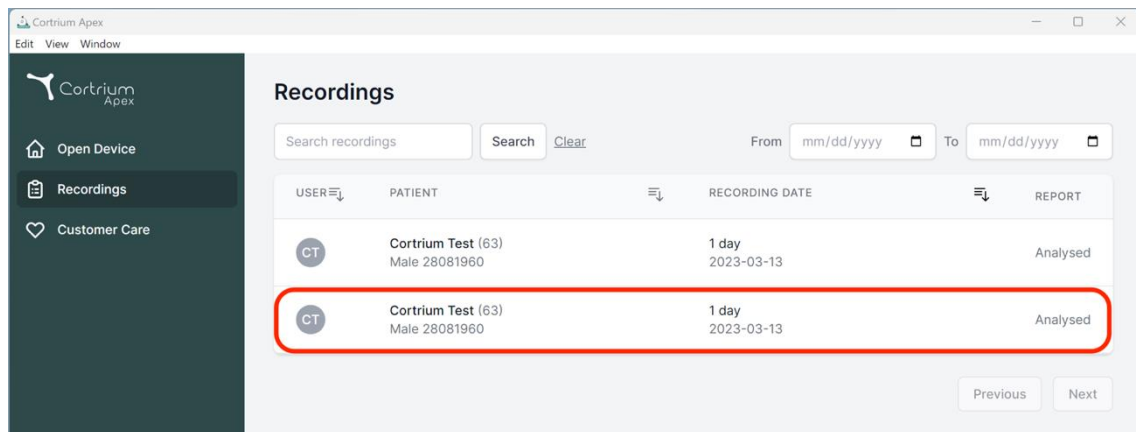
Features ordered: HRV AV-block Fast Track

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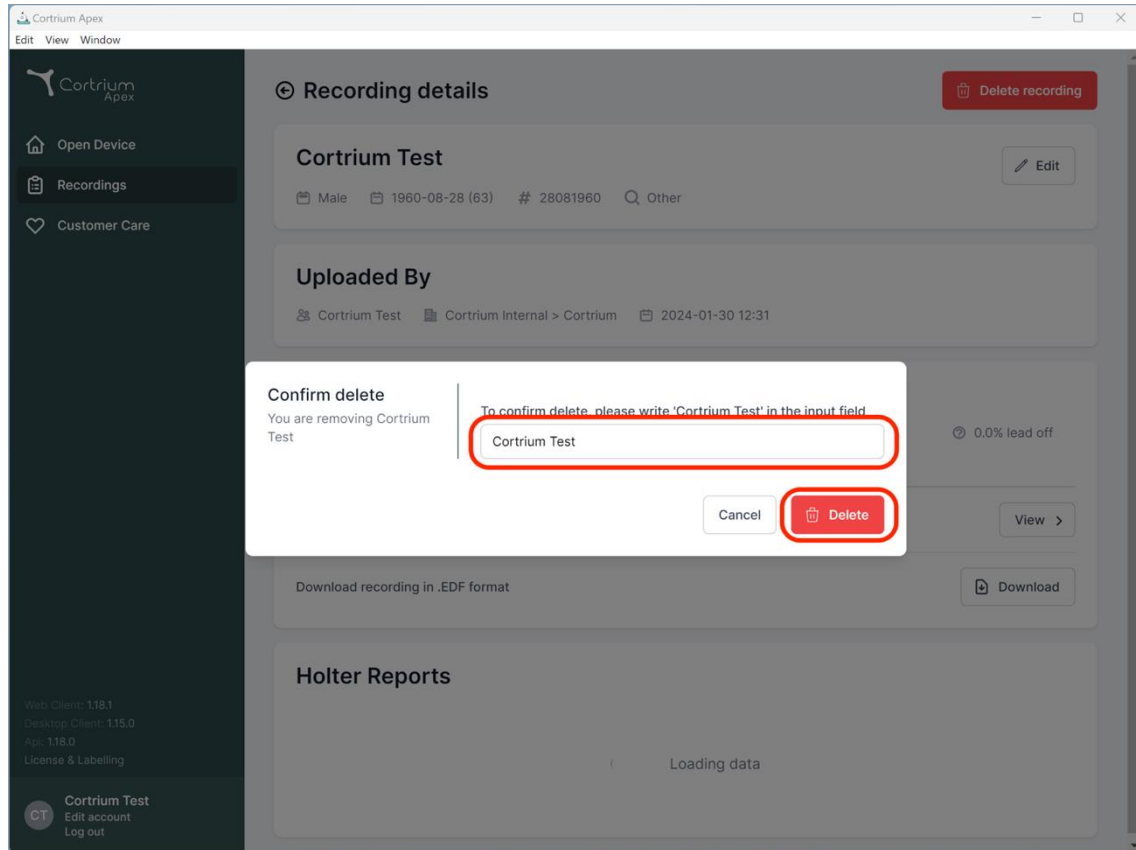
The screenshot shows the Cortrium Apex web application interface. A 'Confirm delete' dialog box is open in the foreground, overlaid on the 'Recording details' page. The dialog box contains the following text and elements:

- Confirm delete**
- You are removing Cortrium Test
- To confirm delete, please write 'Cortrium Test' in the input field
- An input field containing the text 'Cortrium Test' (highlighted with a red circle).
- Buttons for 'Cancel' and 'Delete' (the 'Delete' button is highlighted with a red circle).

The background page, 'Recording details', shows the following information:

- Recording details** (with a 'Delete recording' button)
- Cortrium Test** (with an 'Edit' button)
- Male, 1960-08-28 (63), # 28081960, Other
- Uploaded By**
- Cortrium Test, Cortrium Internal > Cortrium, 2024-01-30 12:31
- 0.0% lead off
- Download recording in .EDF format (with a 'Download' button)
- Holter Reports**
- Loading data

The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. At the bottom left, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The bottom right corner of the page shows '21 / 34'.

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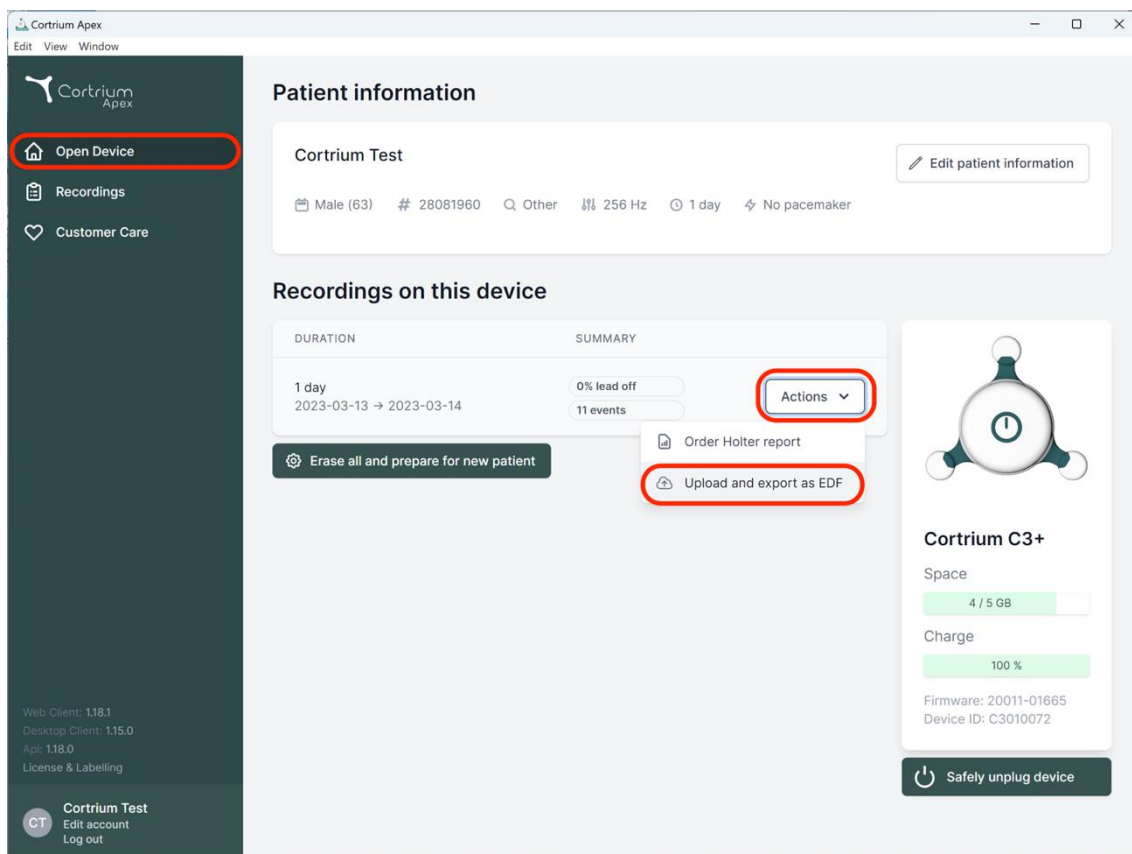
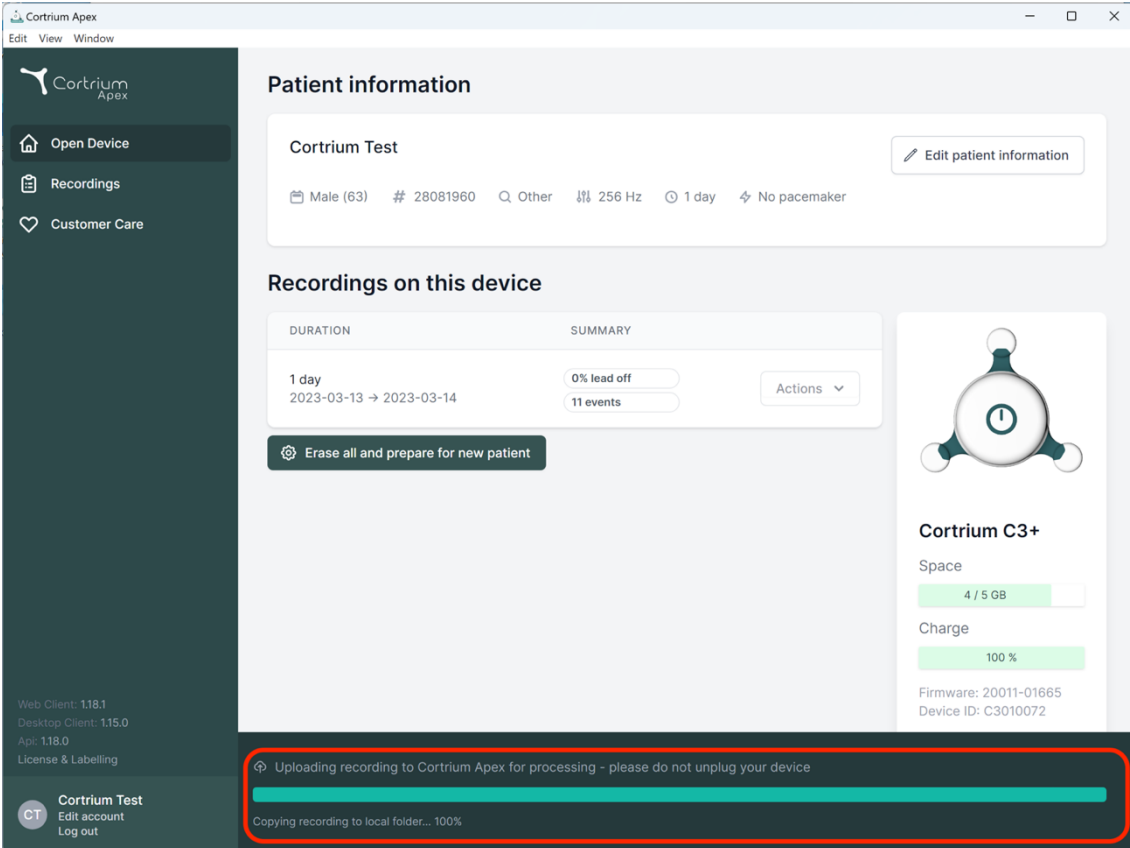
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

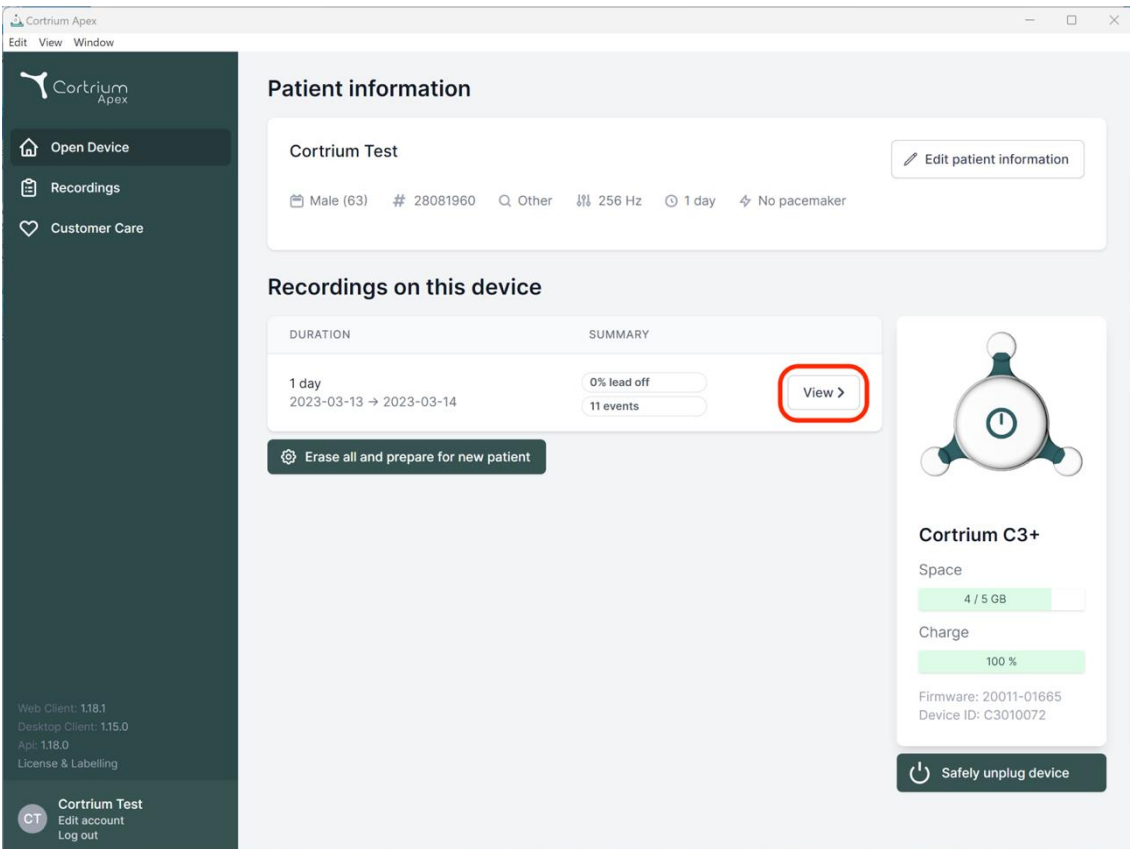
[Erase all and prepare for new patient](#)

Uploading recording to Cortrium Apex for processing - please do not unplug your device

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Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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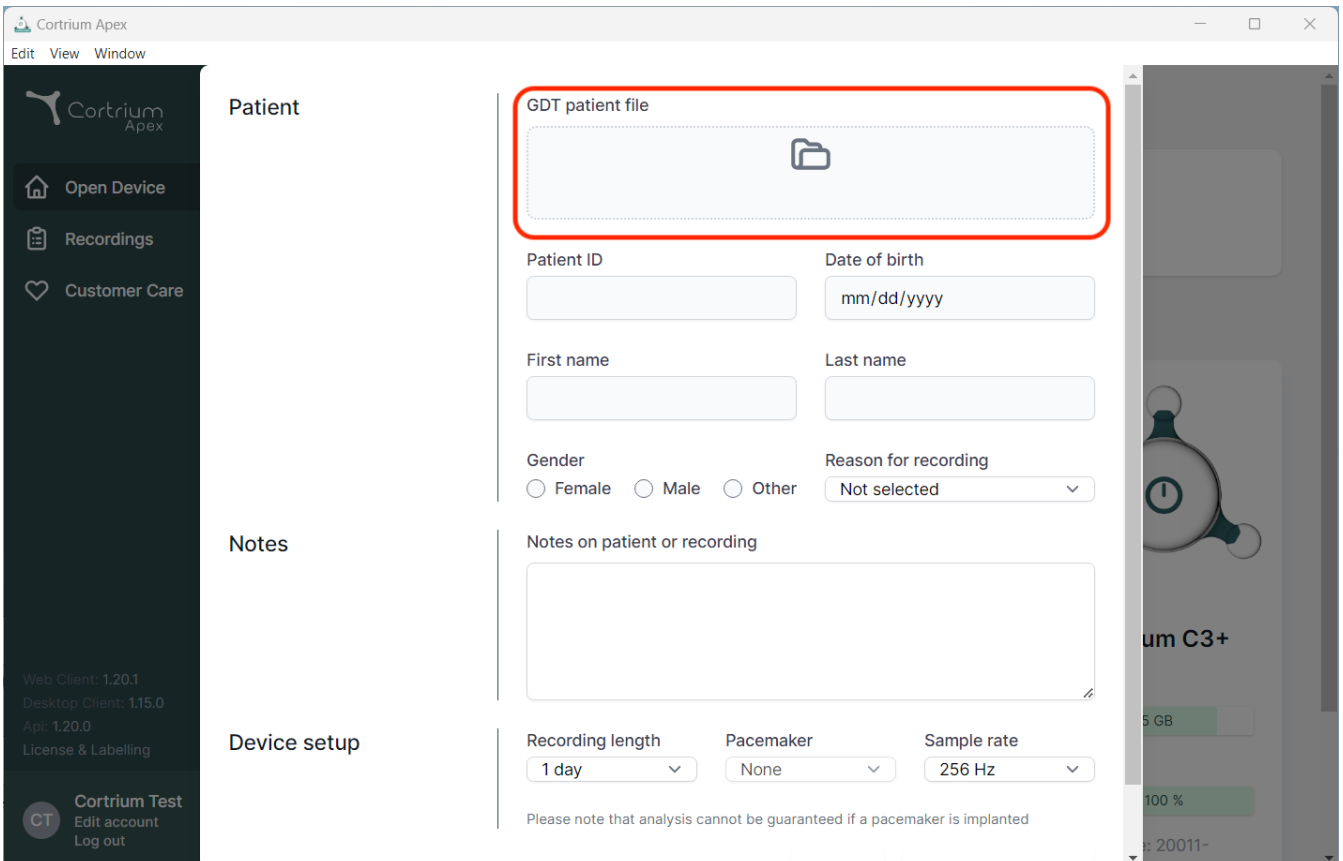
The screenshot shows the Cortrium Apex web application interface. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section contains a form for entering patient information. The 'GDT patient file' field is highlighted with a red box. Below it are fields for Patient ID, Date of birth (mm/dd/yyyy), First name, Last name, Gender (Female, Male, Other), and Reason for recording (Not selected). The Notes section has a text area for 'Notes on patient or recording'. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. The left sidebar contains navigation options: Open Device, Recordings, Customer Care, and user information for Cortrium Test.

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Web Client: 1.20.1
Desktop Client: 1.15.0
App: 1.20.0
License & Labelling

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
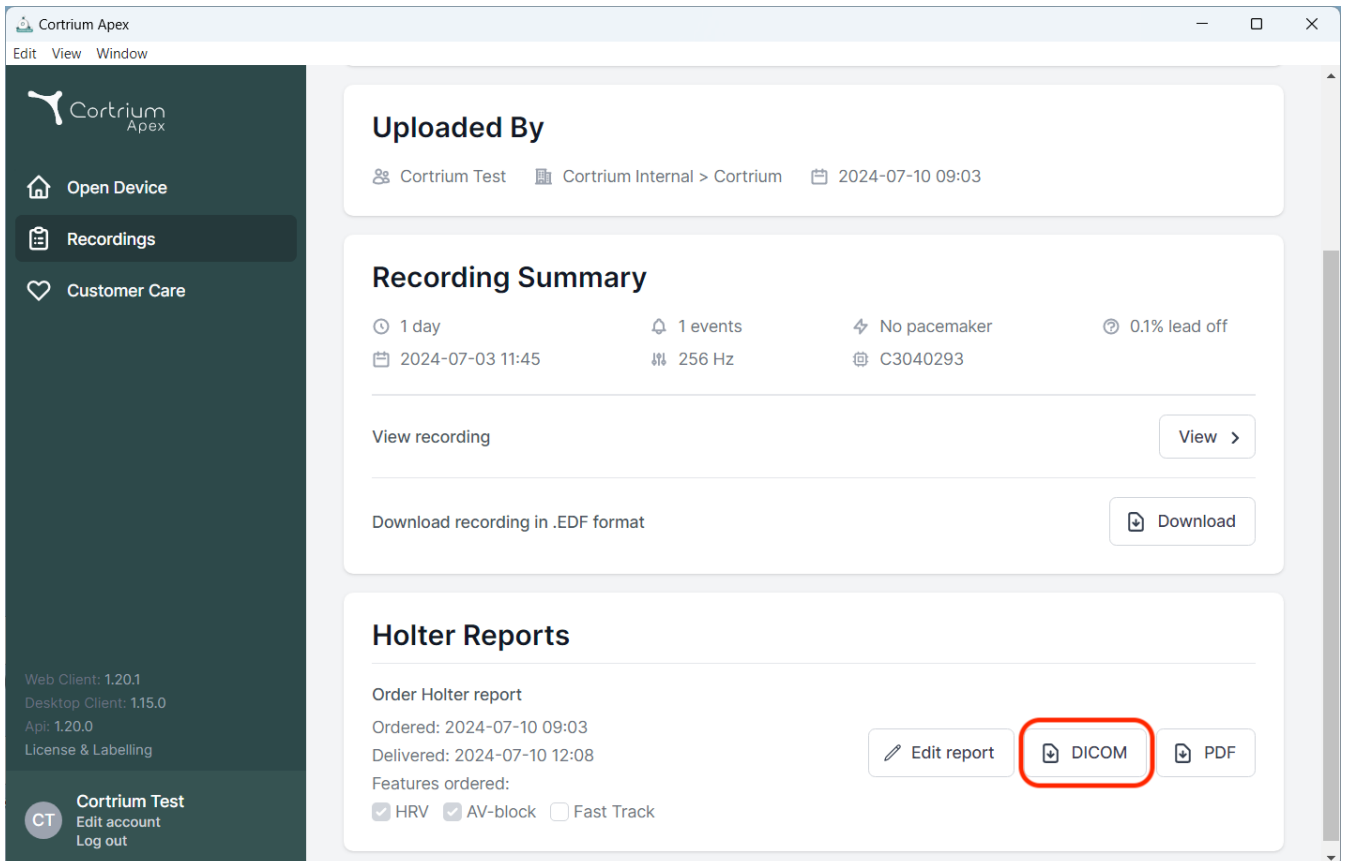
The screenshot shows the Cortrium Apex web interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is divided into three sections:

- Uploaded By:** Shows the user 'Cortrium Test' and the date '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: 1 day duration, 1 event, No pacemaker, 0.1% lead off, and a recording date of 2024-07-03 11:45. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows the report order details: Ordered: 2024-07-10 09:03, Delivered: 2024-07-10 12:08. It lists ordered features: HRV (checked), AV-block (checked), and Fast Track (unchecked). Three buttons are present: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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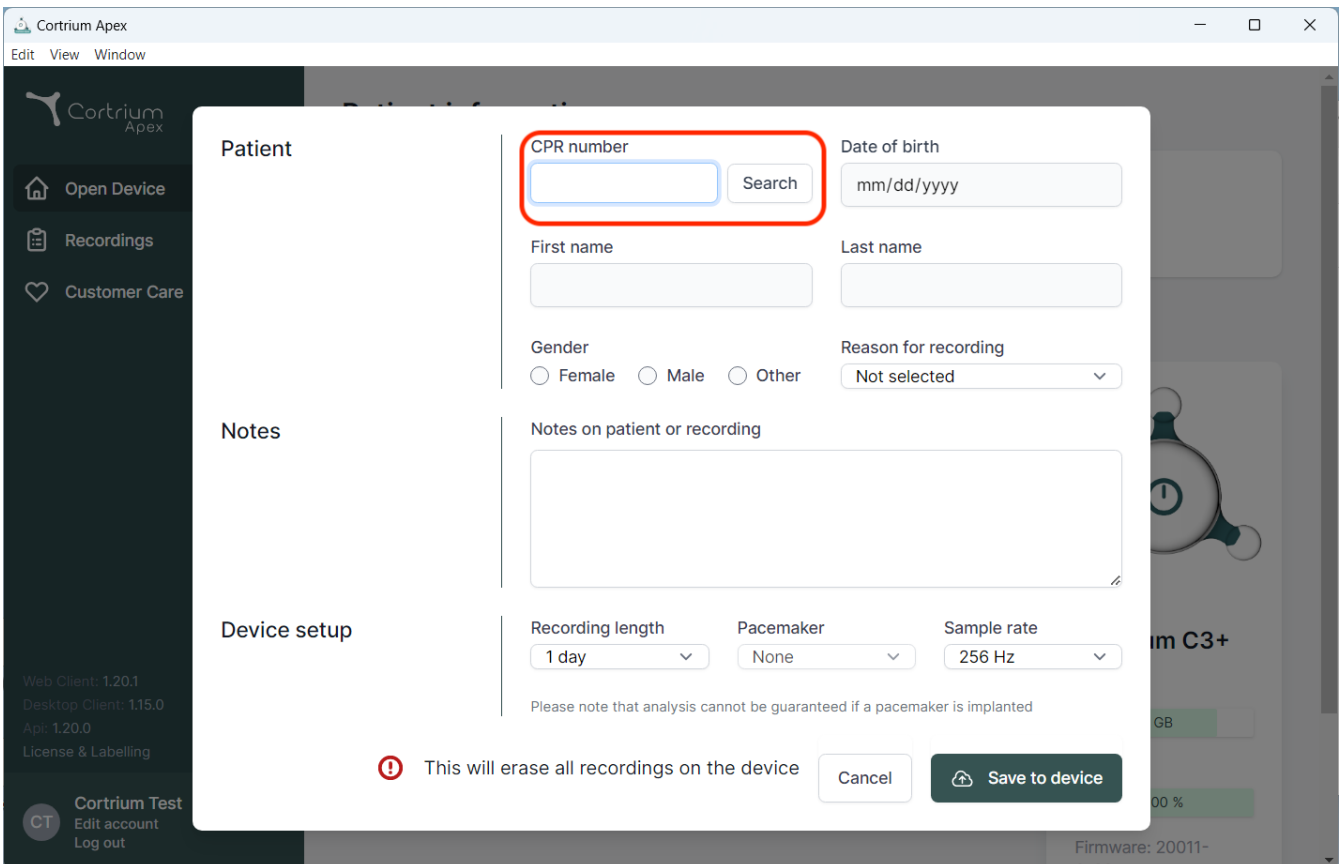Recordings

Customer Care

Web Client: 1.20.1
Desktop Client: 1.15.0
App: 1.20.0
License & Labelling

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! This will erase all recordings on the device

Cancel

09/2024

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danmark
E-post: info@cortrium.com • Nettsted: www.cortrium.com

Cortrium Apex UDI(01)05745000379033