Cortrium Apex 1.20

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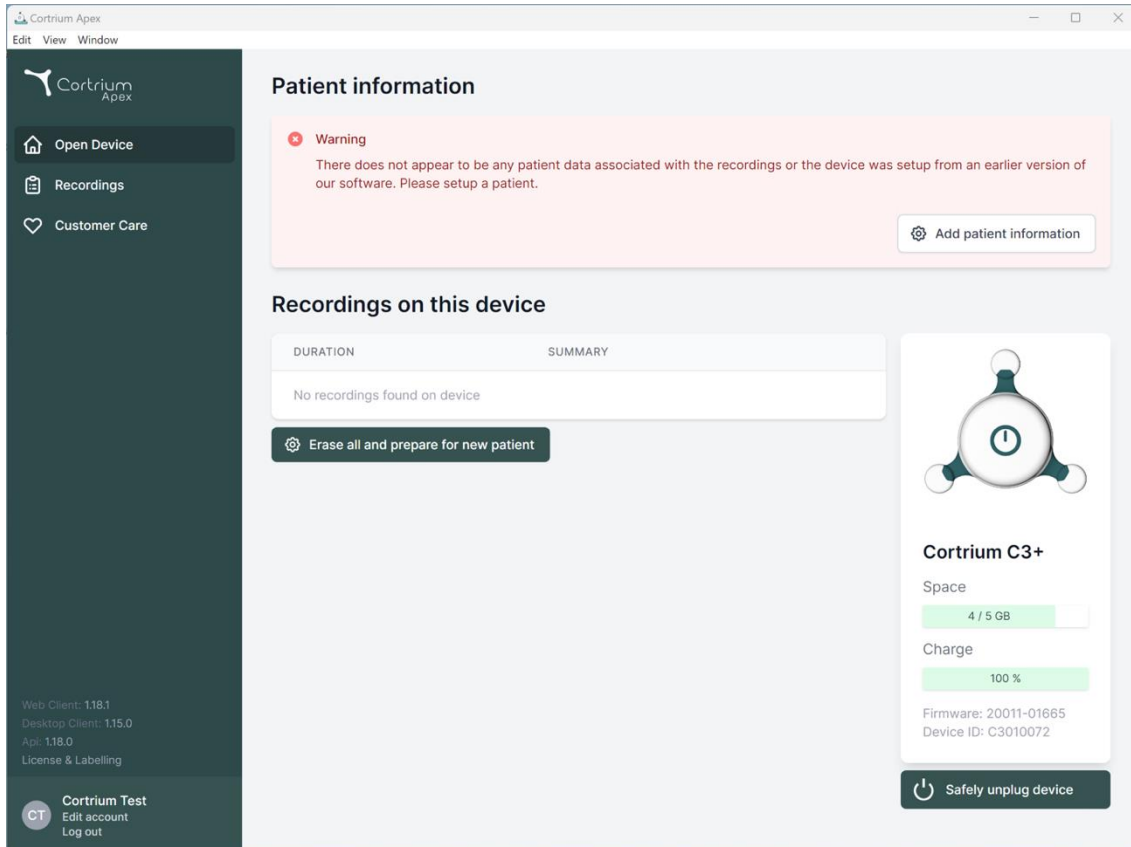
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, there are version numbers (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, License & Labelling) and a user profile section for 'Cortrium Test' with 'Edit account' and 'Log out' options.

The main content area is titled 'Patient information' and features a pink warning box: 'Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient.' with an 'Add patient information' button.

Below this is the 'Recordings on this device' section, which contains a table with columns 'DURATION' and 'SUMMARY'. The table is empty, showing 'No recordings found on device'. A button 'Erase all and prepare for new patient' is located below the table.

On the right side, there is a device status card for 'Cortrium C3+'. It displays a battery icon, 'Space' usage (4 / 5 GB), 'Charge' level (100%), and technical details: 'Firmware: 20011-01665' and 'Device ID: C3010072'. A 'Safely unplug device' button is at the bottom of the card.

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Cortrium Apex
Edit View Window

Cortrium Apex

Open Device
Recordings
Customer Care

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: Female Male Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

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⚠ This will erase all recordings on the device Cancel **Save to device**

Cortrium C3+
pace
4 / 5 GB
charge
100 %
firmware: 20011-01665
device ID: C3010072

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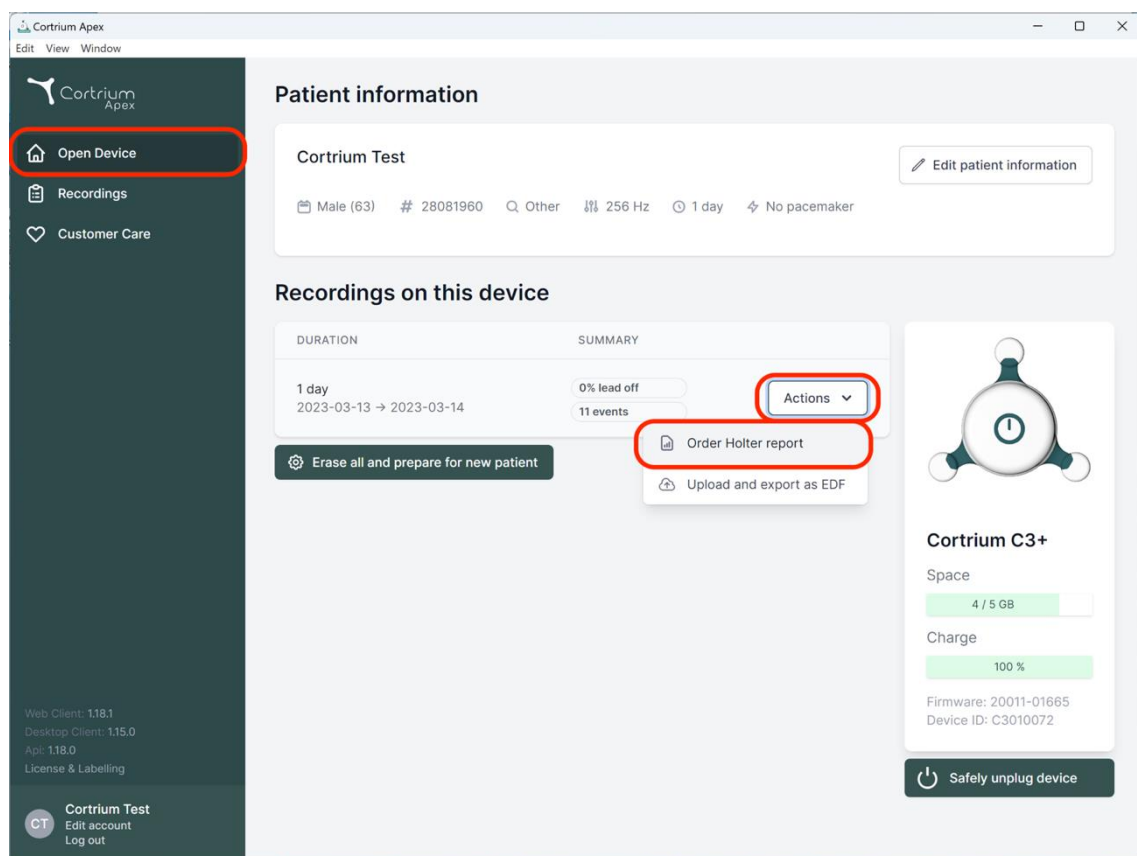
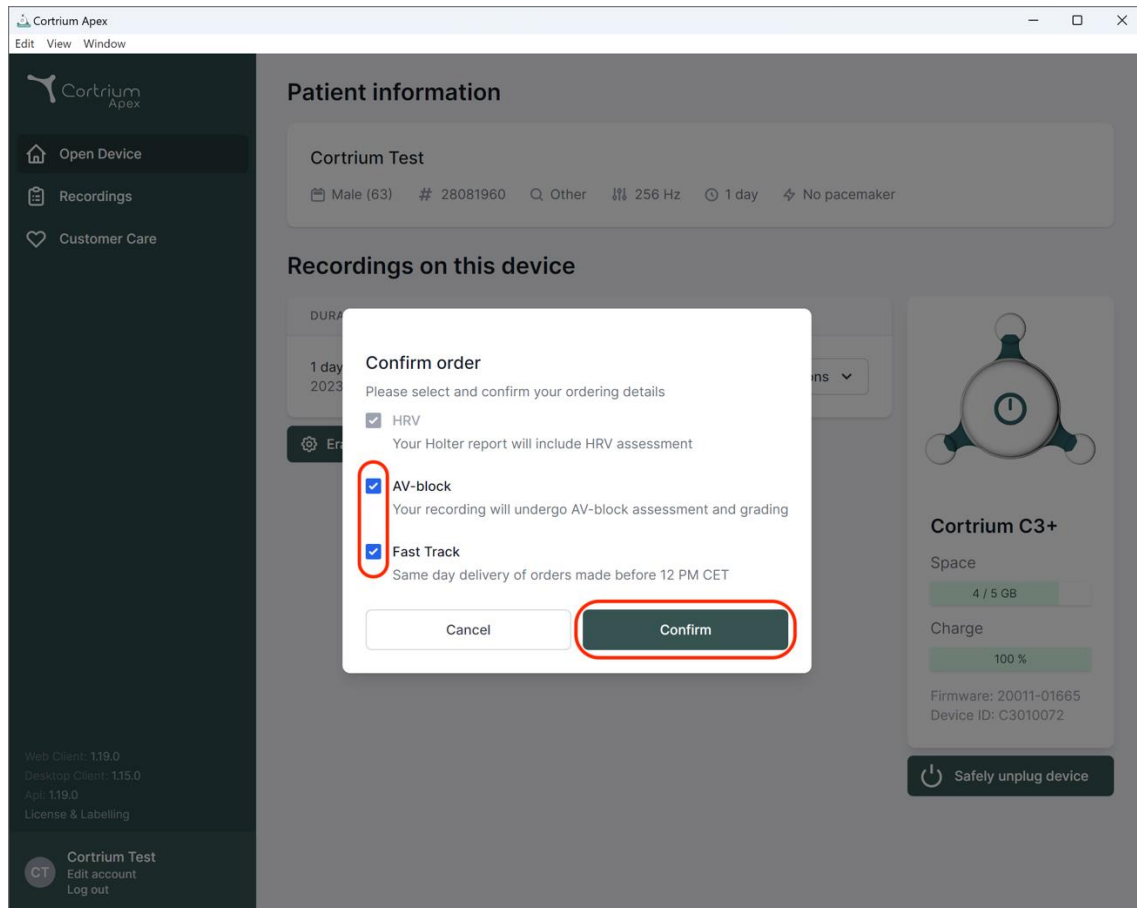
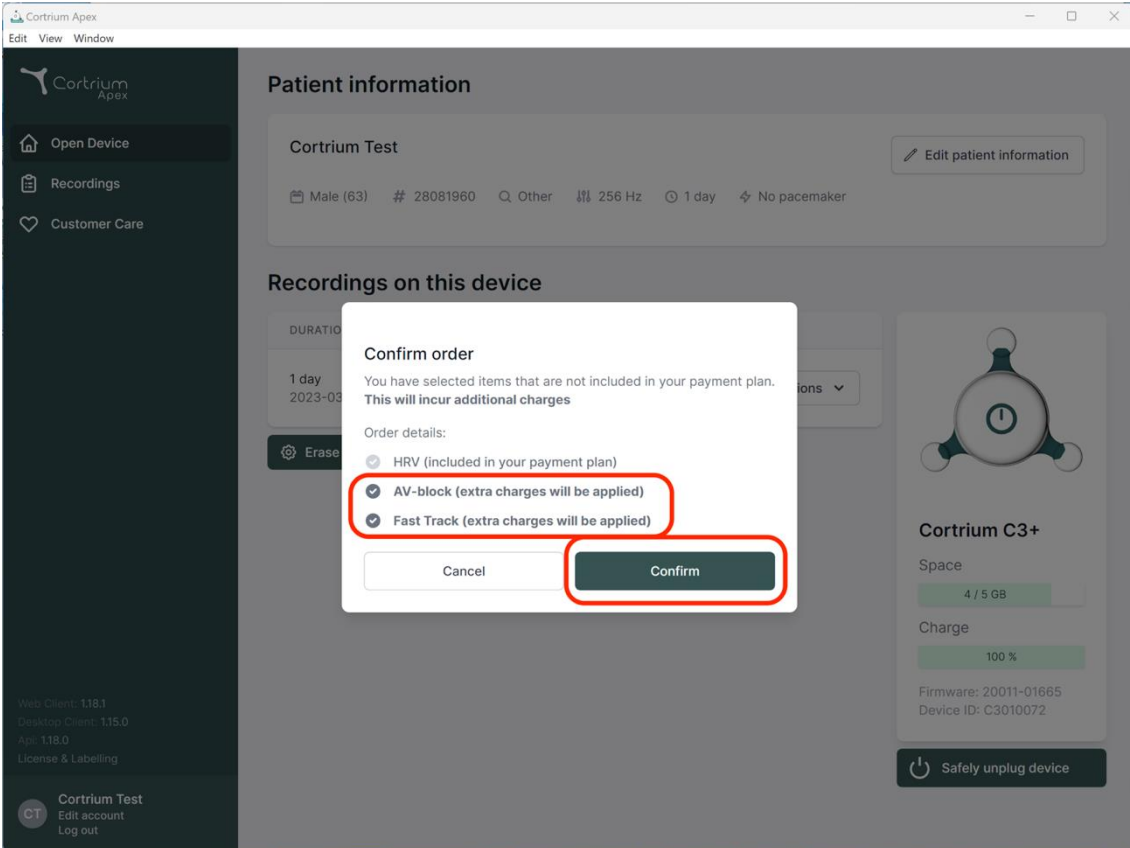
Erase

Confirm order

You have selected items that are not included in your payment plan.
This will incur additional charges

Order details:

- HRV (included in your payment plan)
- AV-block (extra charges will be applied)
- Fast Track (extra charges will be applied)

Cancel Confirm

Cortrium C3+

Space

4 / 5 GB

Charge

100 %

Firmware: 20011-01665


Device ID: C3010072

Safely unplug device

Web Client: 118.1
Desktop Client: 115.0
App: 118.0
License & Labelling

Cortrium Test
CT Edit account
Log out

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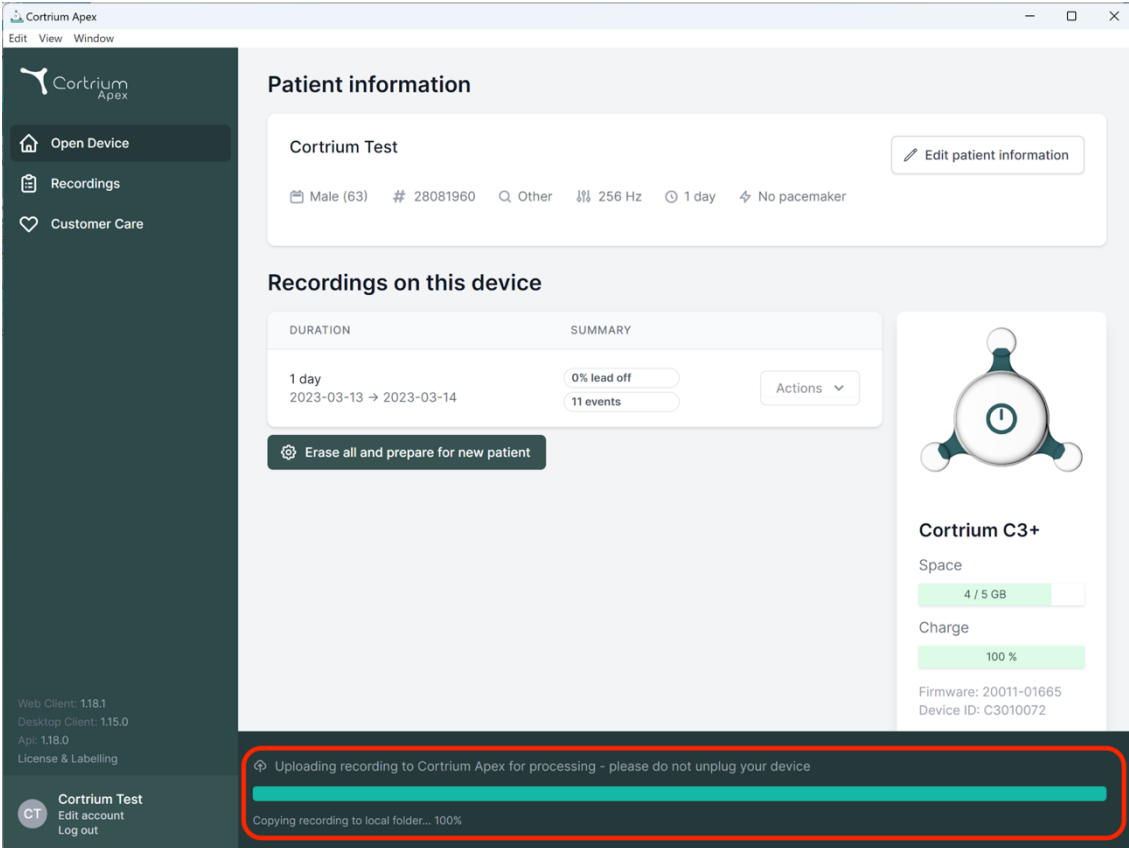The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and test duration (1 day, 2023-03-13 to 2023-03-14). Below this is a 'Recordings on this device' section with a table showing a 1-day recording with 0% lead off and 11 events. A 'Cortrium C3+' device status panel on the right shows 4/5 GB space and 100% charge. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'.

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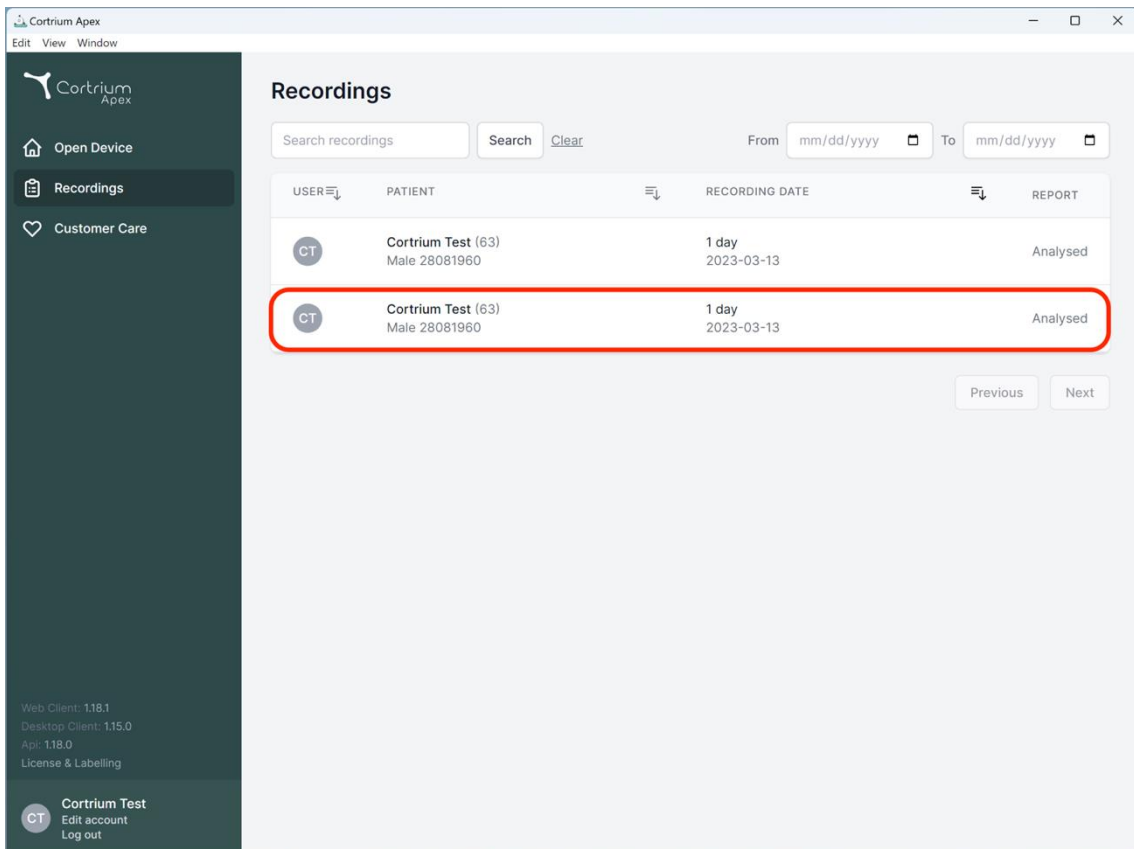
The screenshot displays the 'Recordings' section of the Cortrium Apex web application. The interface includes a search bar, a date range selector, and a table of recording entries. The table has the following structure:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

The second row of the table is highlighted with a red border. Below the table are 'Previous' and 'Next' navigation buttons. The left sidebar contains navigation options: 'Open Device', 'Recordings', and 'Customer Care'. At the bottom of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. The footer of the sidebar lists version information: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'App: 1.18.0', and 'License & Labelling'.

Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

View recording View >

Download recording in .EDF format Download

Holter Reports

Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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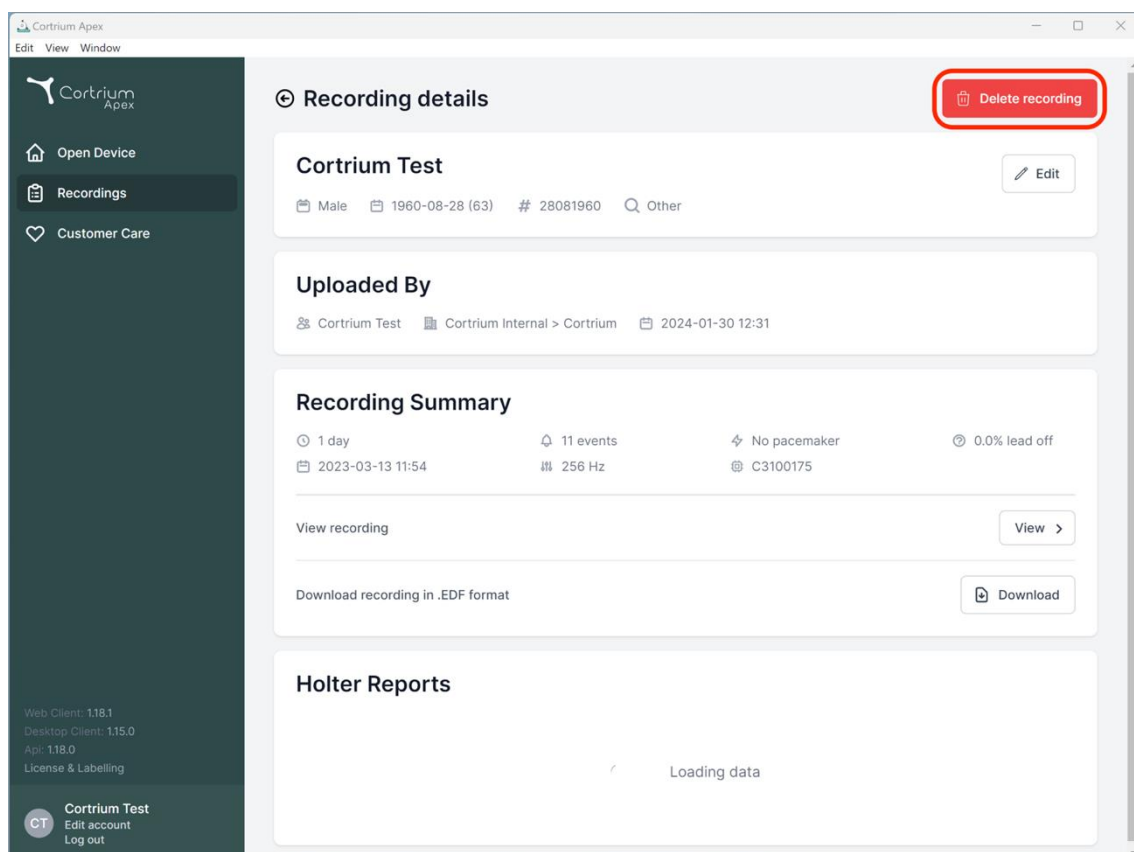
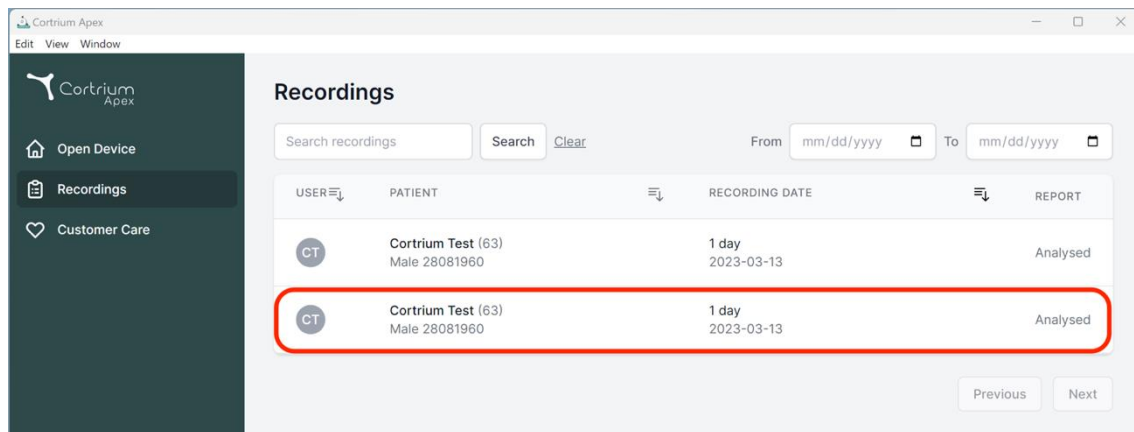
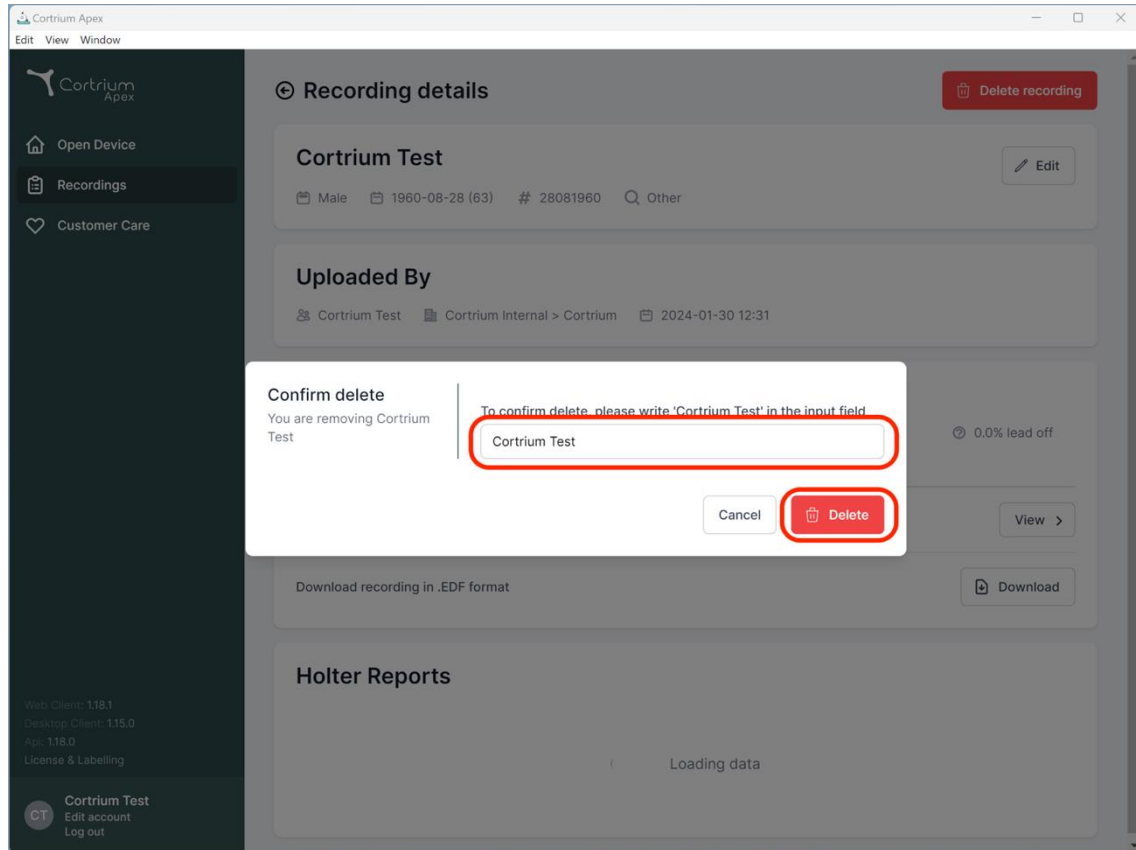
The screenshot shows the Cortrium Apex web application interface. A 'Confirm delete' dialog box is open in the foreground, overlaid on the 'Recording details' page. The dialog box contains the following text and elements:

- Confirm delete**
- You are removing Cortrium Test
- To confirm delete, please write 'Cortrium Test' in the input field
- An input field containing the text 'Cortrium Test' (highlighted with a red circle).
- Buttons for 'Cancel' and 'Delete' (the 'Delete' button is highlighted with a red circle).

The background page, 'Recording details', shows the following information:

- Recording details** (with a 'Delete recording' button)
- Cortrium Test** (with an 'Edit' button)
- Male, 1960-08-28 (63), # 28081960, Other
- Uploaded By**
- Cortrium Test, Cortrium Internal > Cortrium, 2024-01-30 12:31
- 0.0% lead off
- Download recording in .EDF format (with a 'Download' button)
- Holter Reports**
- Loading data

The left sidebar contains navigation options: Open Device, Recordings, and Customer Care. The bottom left corner shows the user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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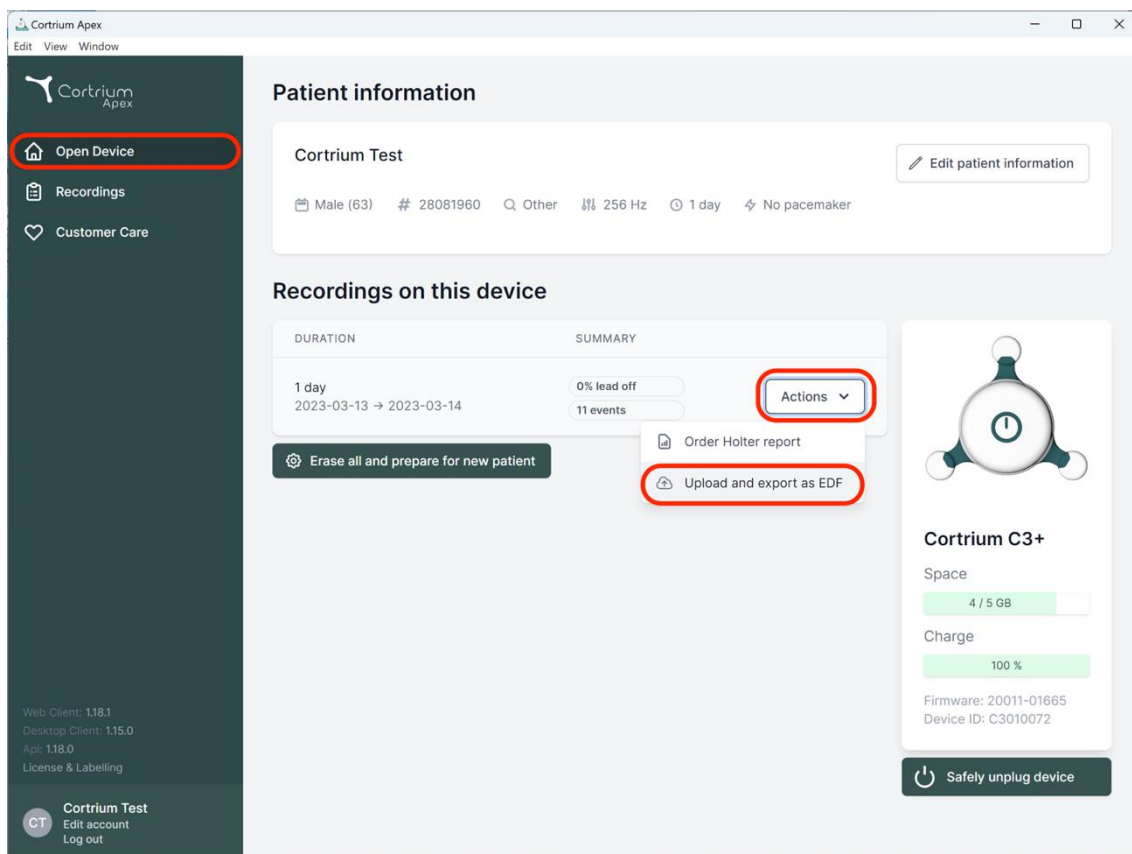
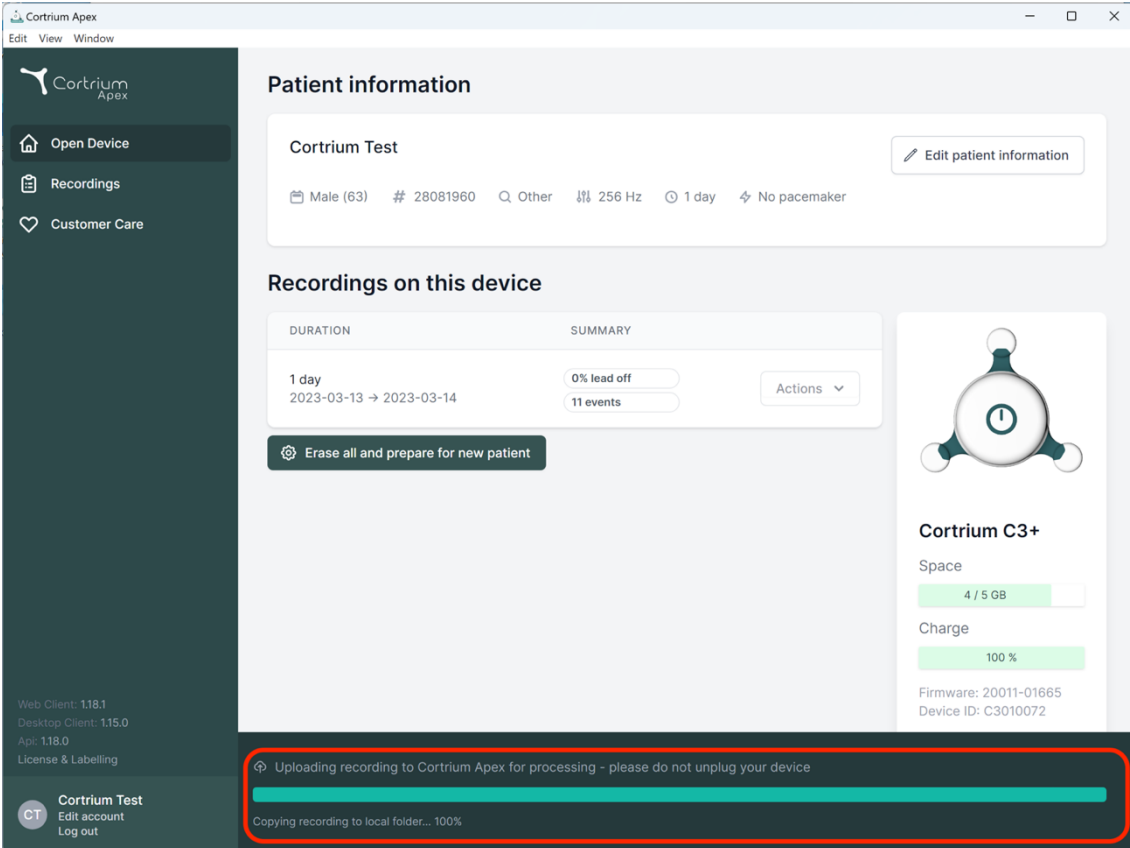
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	Actions

[Erase all and prepare for new patient](#)

Uploading recording to Cortrium Apex for processing - please do not unplug your device

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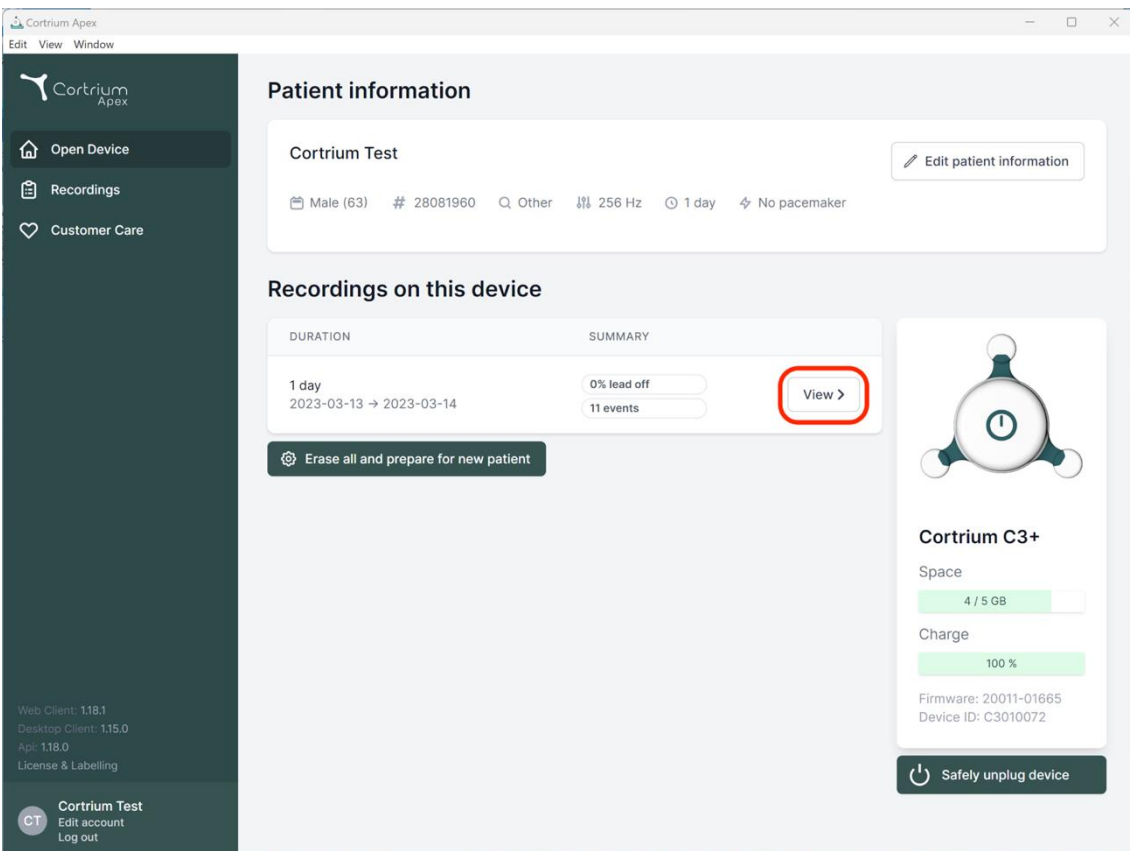
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
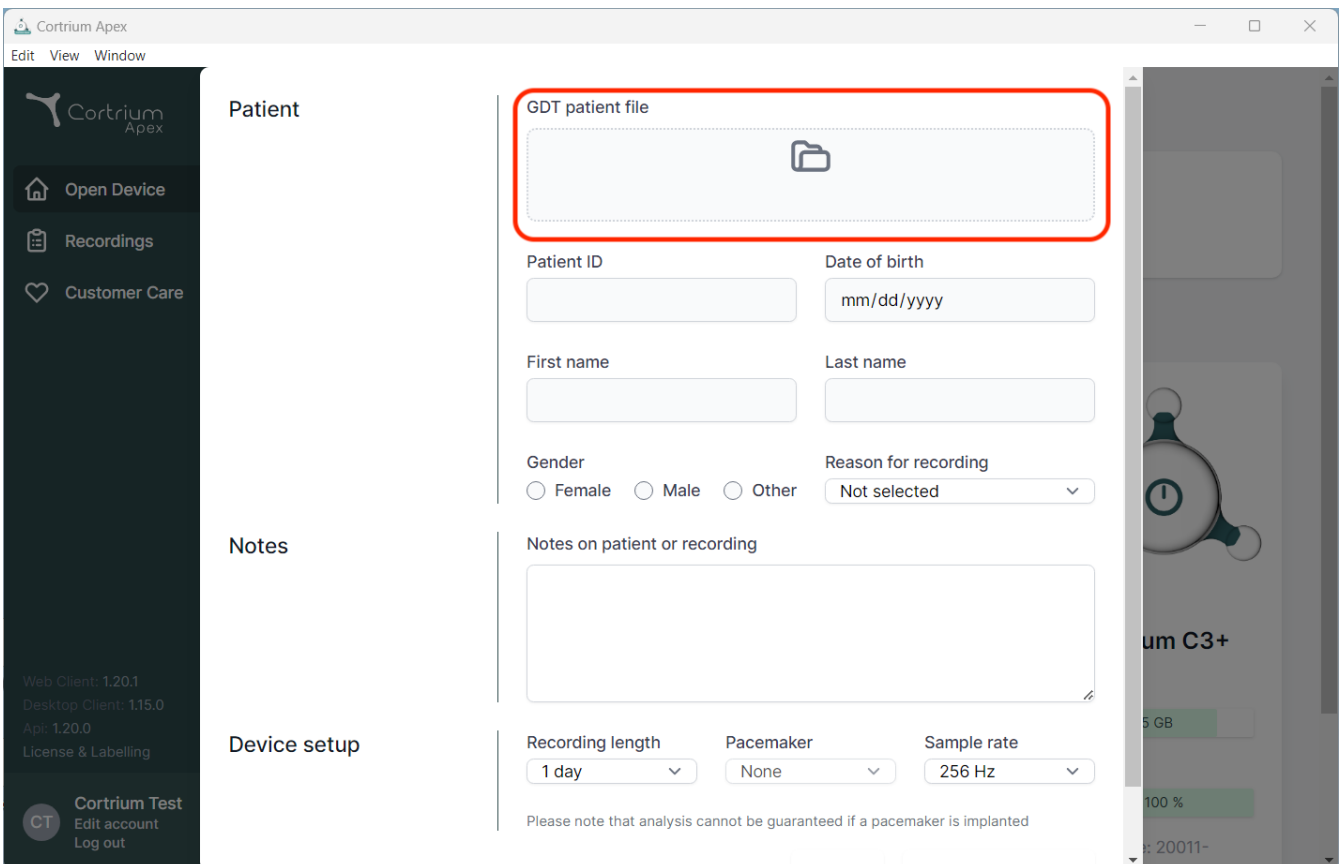
The screenshot displays the Cortrium Apex web application interface. The main content area is divided into three sections: Patient, Notes, and Device setup. The Patient section is highlighted with a red box around the 'GDT patient file' upload area, which contains a folder icon. Below this are input fields for Patient ID, Date of birth (mm/dd/yyyy), First name, Last name, Gender (Female, Male, Other), and Reason for recording (Not selected). The Notes section has a text area for 'Notes on patient or recording'. The Device setup section includes dropdown menus for Recording length (1 day), Pacemaker (None), and Sample rate (256 Hz). A disclaimer at the bottom states: 'Please note that analysis cannot be guaranteed if a pacemaker is implanted'. The left sidebar contains navigation options: Open Device, Recordings, Customer Care, and user information for Cortrium Test.

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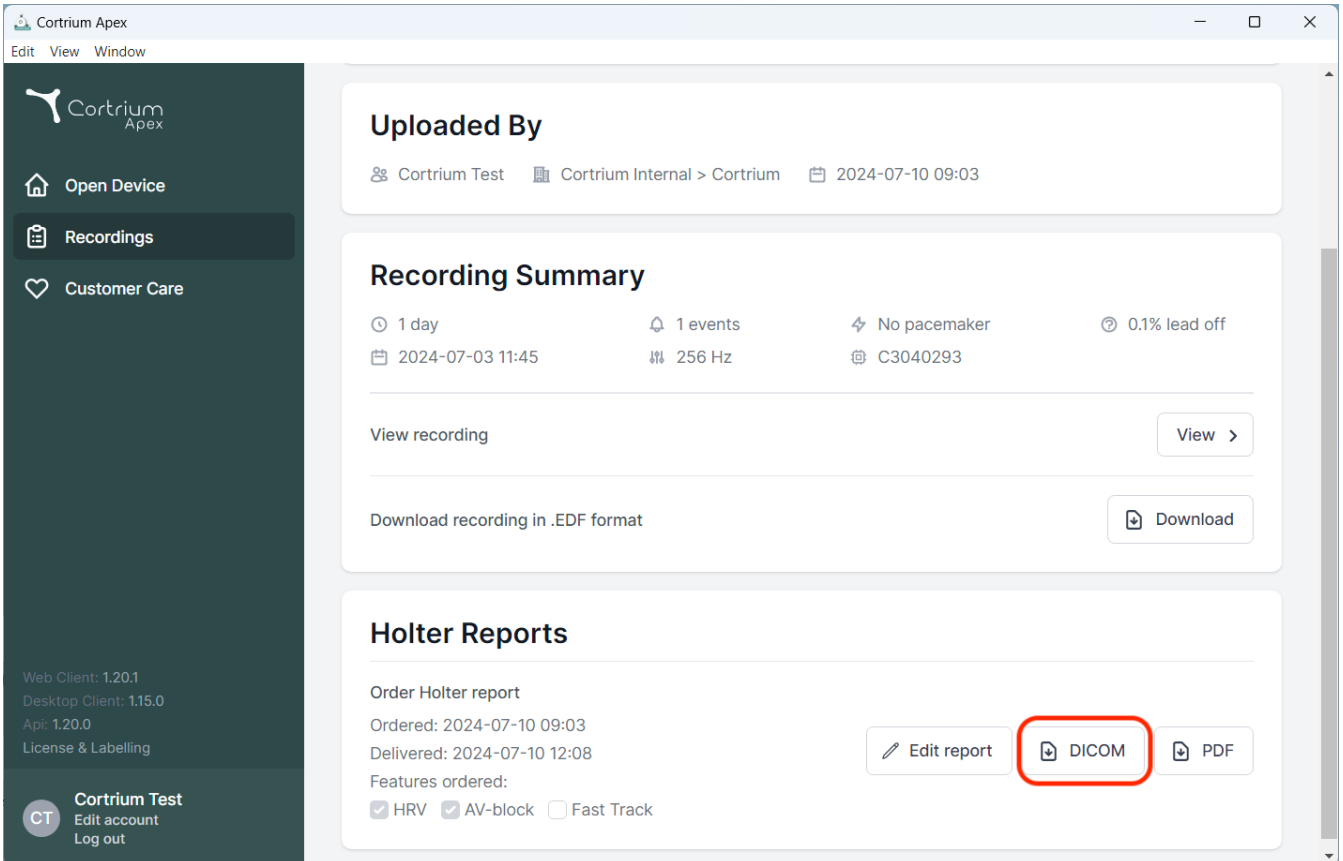
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' for 'Cortrium Internal > Cortrium' on '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' duration, '2024-07-03 11:45' start time, '1 events', '256 Hz' rate, 'No pacemaker', and '0.1% lead off'. It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists ordered features: 'HRV' (checked), 'AV-block' (checked), and 'Fast Track' (unchecked). Three buttons are visible: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'. Version information for Web Client (1.20.1), Desktop Client (1.15.0), and Api (1.20.0) is also present.

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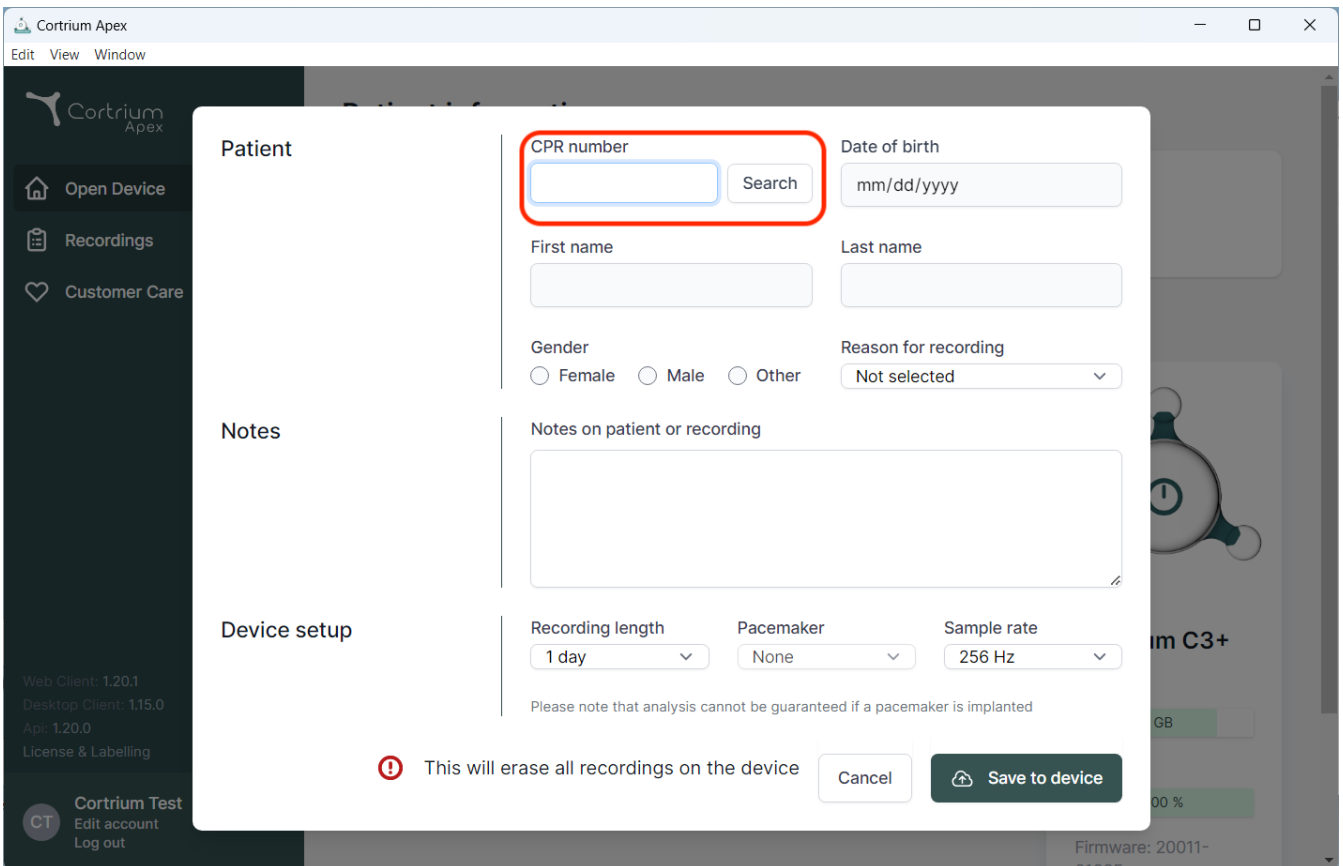Recordings

Customer Care

Web Client: 1.20.1
Desktop Client: 1.15.0
App: 1.20.0
License & Labelling

Cortrium Test
Edit account
Log out

Firmware: 20011-01000

00 %

GB

im C3+

Patient

CPR number Search

Date of birth

First name

Last name

Gender
 Female Male Other

Reason for recording
Not selected

Notes

Notes on patient or recording

Device setup

Recording length

Pacemaker

Sample rate

Please note that analysis cannot be guaranteed if a pacemaker is implanted

 This will erase all recordings on the device

Cancel

09/2024

„Cortrium ApS“ • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Danija
El. paštas: info@cortrium.com • Interneto svetainė: www.cortrium.com

Cortrium Apex UDI(01)05745000379033