Cortrium Apex 1.20

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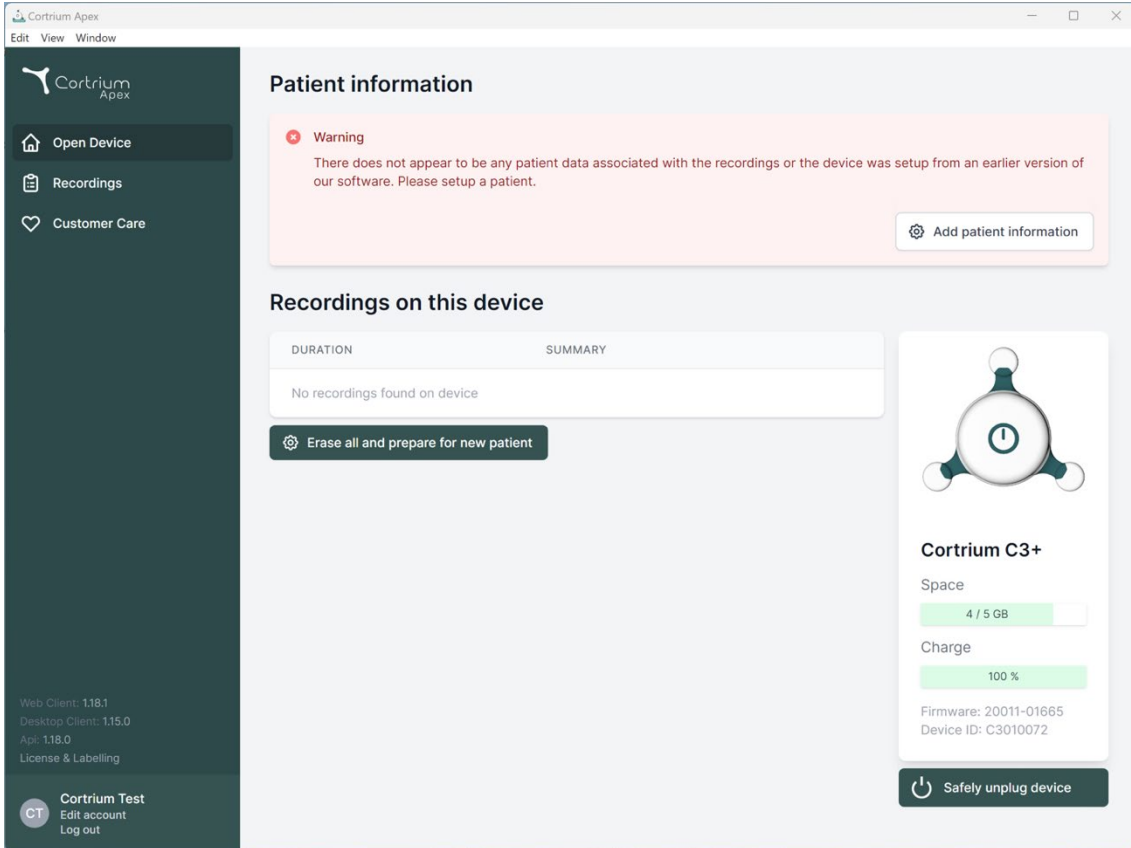
The screenshot shows the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. At the bottom of the sidebar, it shows version information (Web Client: 1.18.1, Desktop Client: 1.15.0, App: 1.18.0, License & Labelling) and user options (Cortrium Test, Edit account, Log out). The main content area is titled "Patient information" and features a warning message: "Warning: There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of our software. Please setup a patient." Below this is a table for "Recordings on this device" with columns for DURATION and SUMMARY, showing "No recordings found on device" and a button to "Erase all and prepare for new patient". On the right, a device status card for "Cortrium C3+" displays "Space: 4 / 5 GB" and "Charge: 100 %", along with "Firmware: 20011-01665" and "Device ID: C3010072". A "Safely unplug device" button is at the bottom of the card.

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Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings
- Customer Care

Patient information

Warning
There does not appear to be any patient data associated with the recordings or the device was setup from an earlier version of

Patient

Patient ID: 28081960
Date of birth: 08/28/1960

First name: Cortrium
Last name: Test

Gender: Female Male Other
Reason for recording: Other

Notes

Notes on patient or recording

Device setup

Recording length: 1 day
Pacemaker: None
Sample rate: 256 Hz

Please note that analysis cannot be guaranteed if a pacemaker is implanted

⚠ This will erase all recordings on the device

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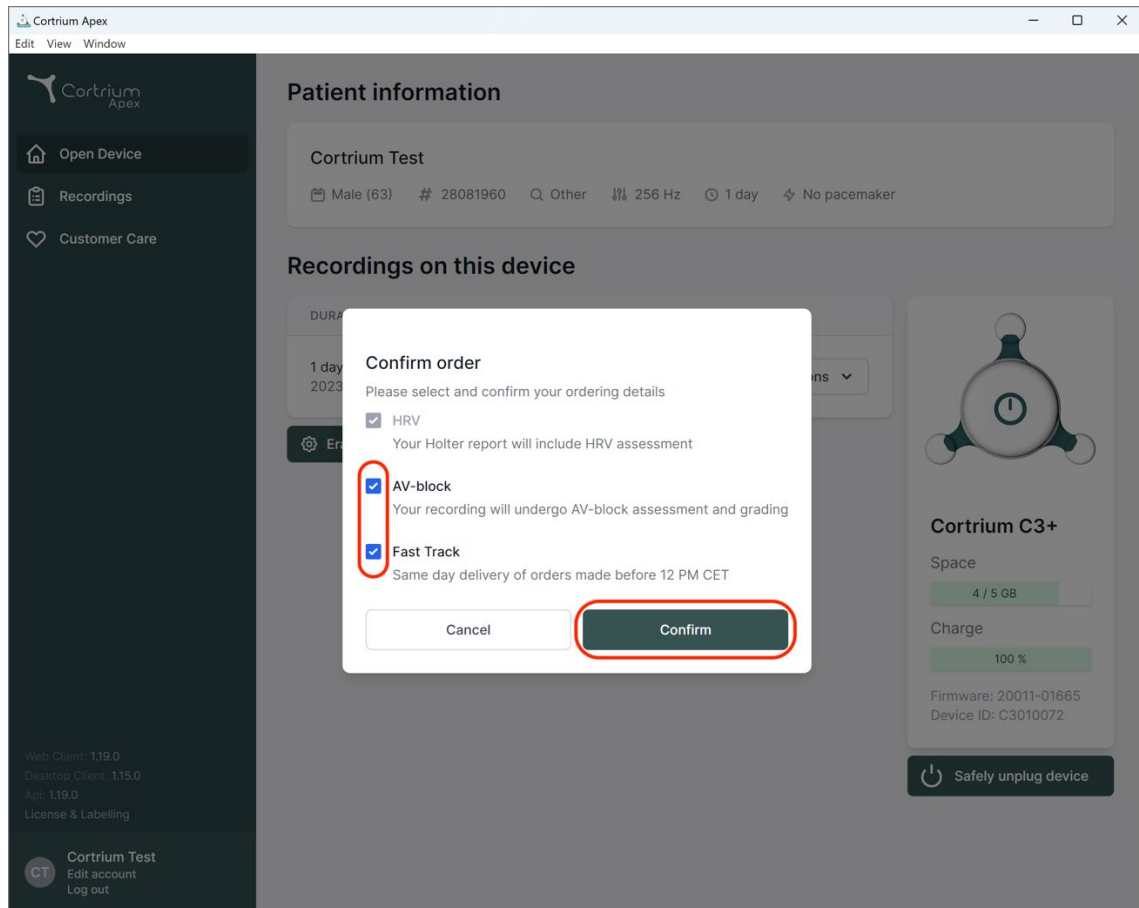
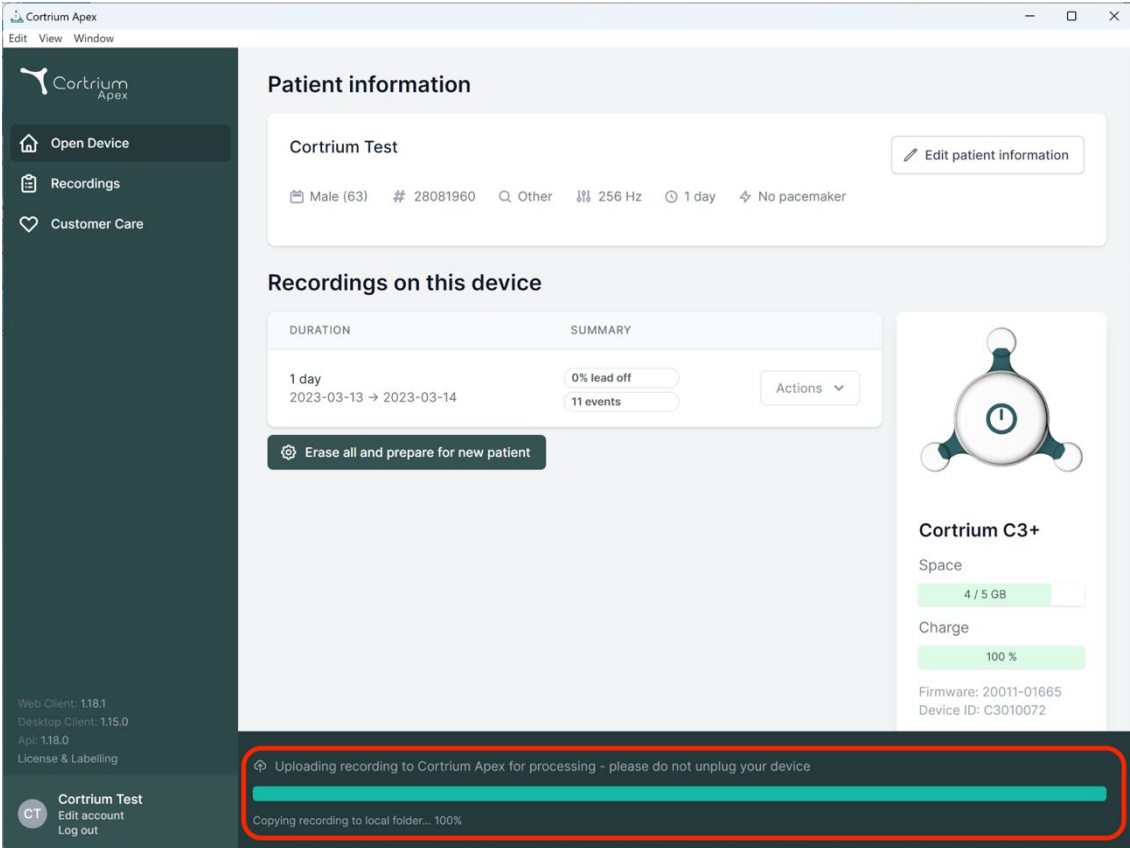
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: Open Device, Recordings, and Customer Care. The main content area is titled 'Patient information' and shows details for 'Cortrium Test', including gender (Male, 63), ID (28081960), and recording duration (1 day, 2023-03-13 to 2023-03-14). Below this, the 'Recordings on this device' section shows a summary of 11 events with a 0% lead off. A 'Cortrium C3+' device status panel on the right indicates 4/5 GB of space used and 100% charge. At the bottom, a red-bordered box highlights a progress bar for 'Uploading recording to Cortrium Apex for processing - please do not unplug your device' and 'Copying recording to local folder... 100%'.

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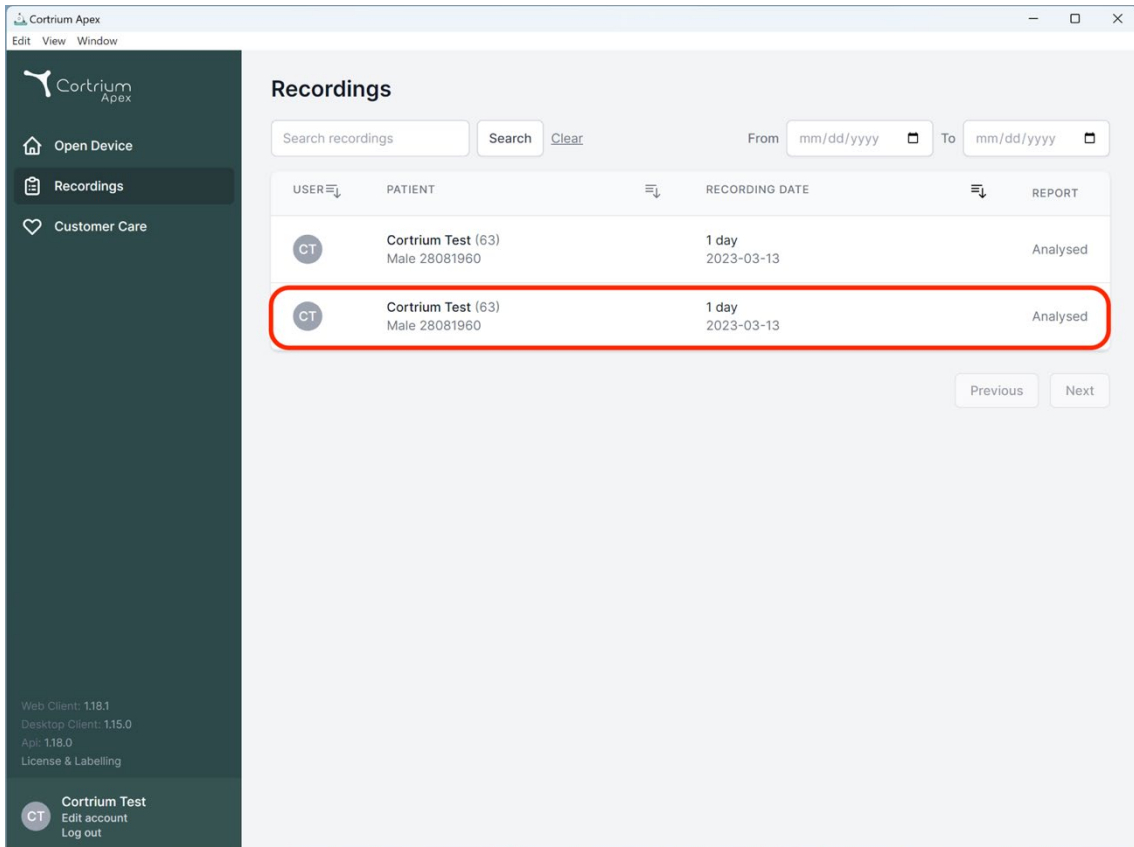
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

Cortrium Apex
Edit View Window

Cortrium Apex

- Open Device
- Recordings**
- Customer Care

Web Client: 1.18.1
Desktop Client: 1.15.0
App: 1.18.0
License & Labelling

CT Cortrium Test
Edit account
Log out

Recording details

Cortrium Test Edit

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

1 day 11 events No pacemaker 0.0% lead off
2023-03-13 11:54 256 Hz C3100175

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Order Holter report

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00

Features ordered:
 HRV AV-block Fast Track

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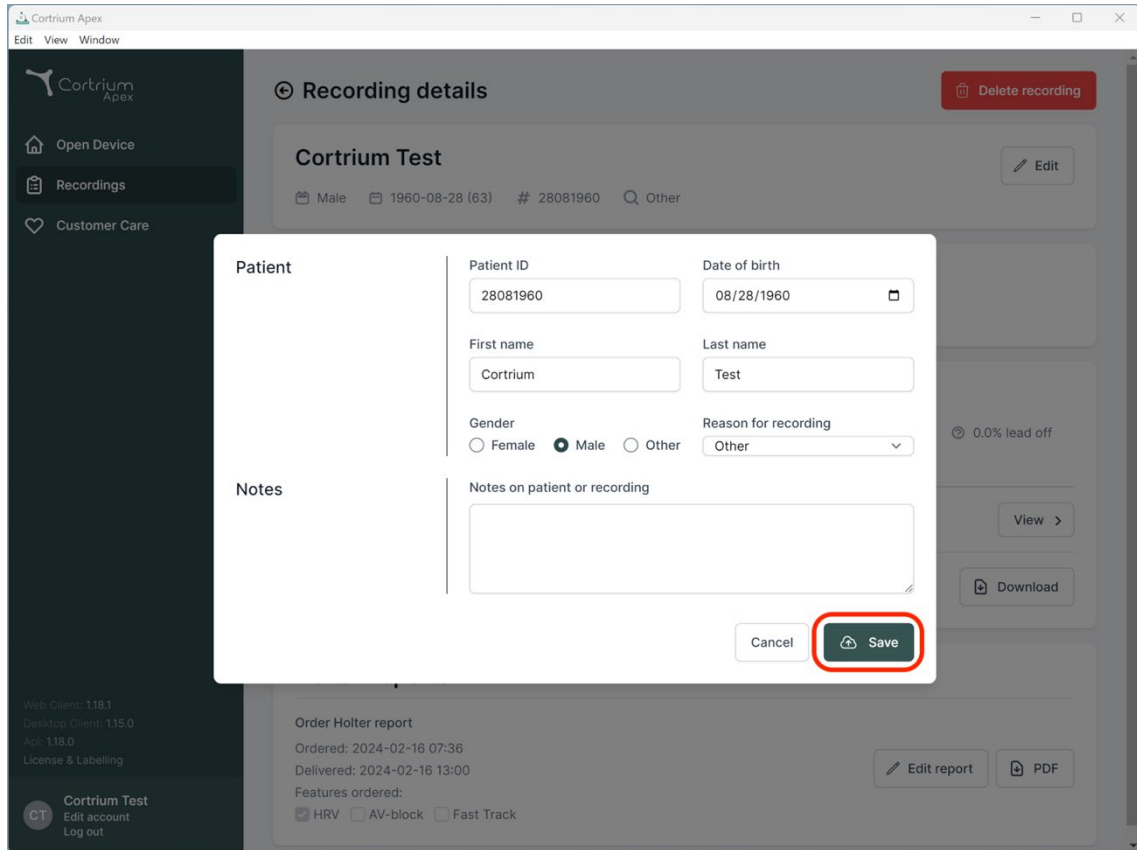
The screenshot shows the 'Recording details' page in the Cortrium Apex application. A modal form is open for editing patient information. The form includes fields for Patient ID (28081960), Date of birth (08/28/1960), First name (Cortrium), Last name (Test), Gender (Male selected), and Reason for recording (Other). A 'Notes' section is also present. The 'Save' button is highlighted with a red circle. The background shows the 'Recording details' page with a 'Delete recording' button and an 'Order Holter report' section.

Recording details

Cortrium Test

Male 1960-08-28 (63) # 28081960 Other

Patient

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: Female Male Other Reason for recording: Other

Notes

Notes on patient or recording

Cancel Save

Web Client: 1.18.1 Desktop Client: 1.15.0 App: 1.18.0 License & Labelling

CT Cortrium Test Edit account Log out

Order Holter report

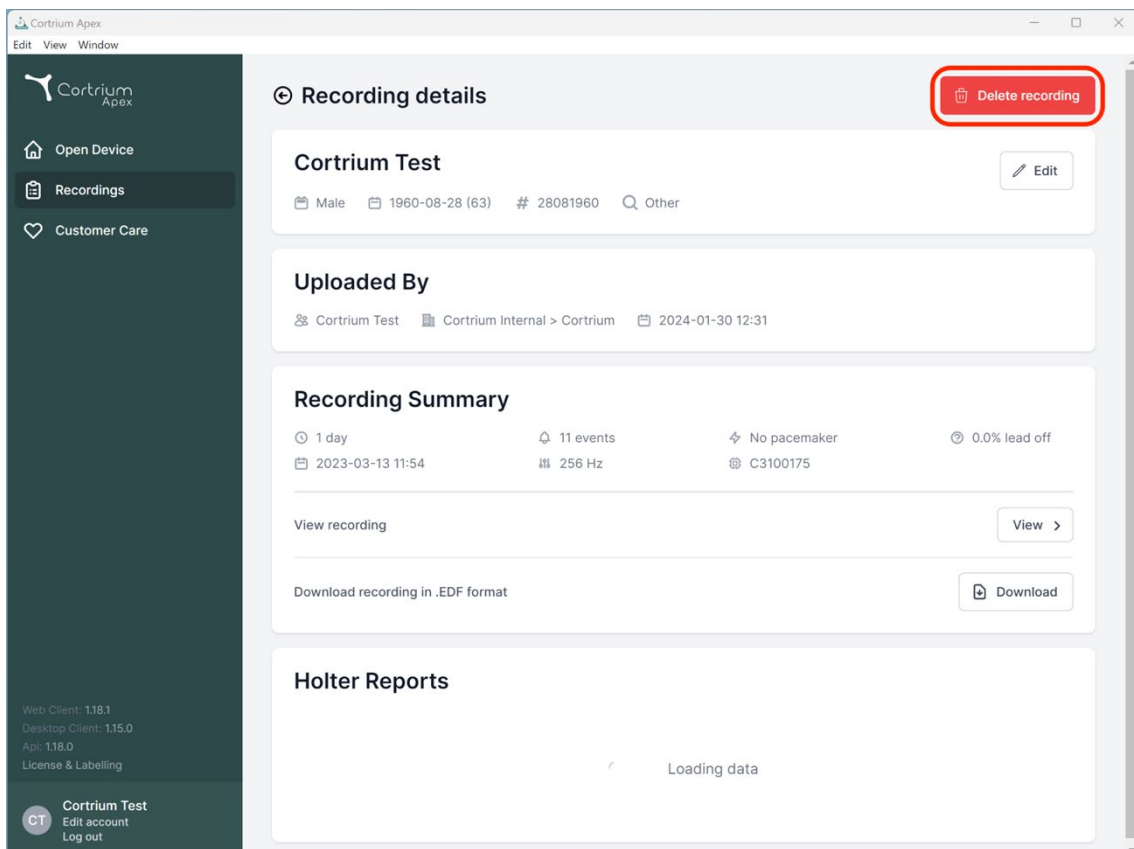
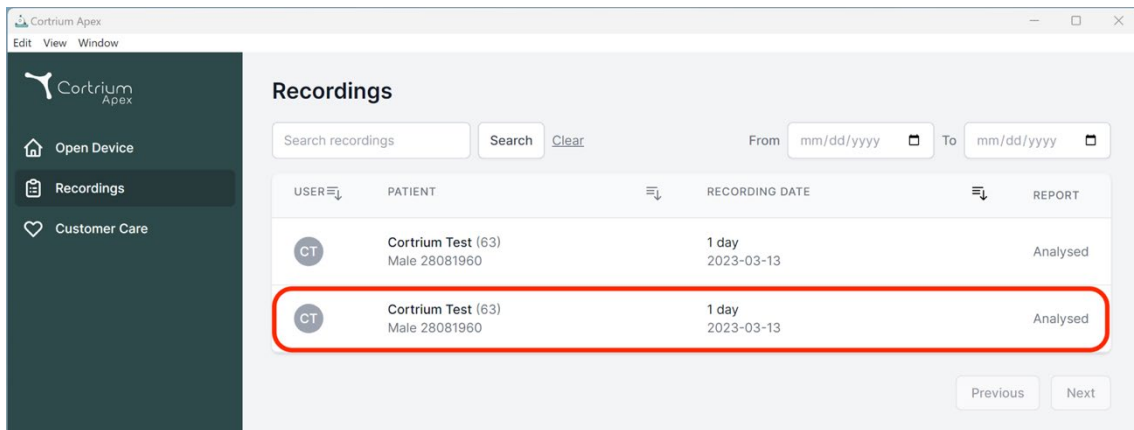
Ordered: 2024-02-16 07:36 Delivered: 2024-02-16 13:00

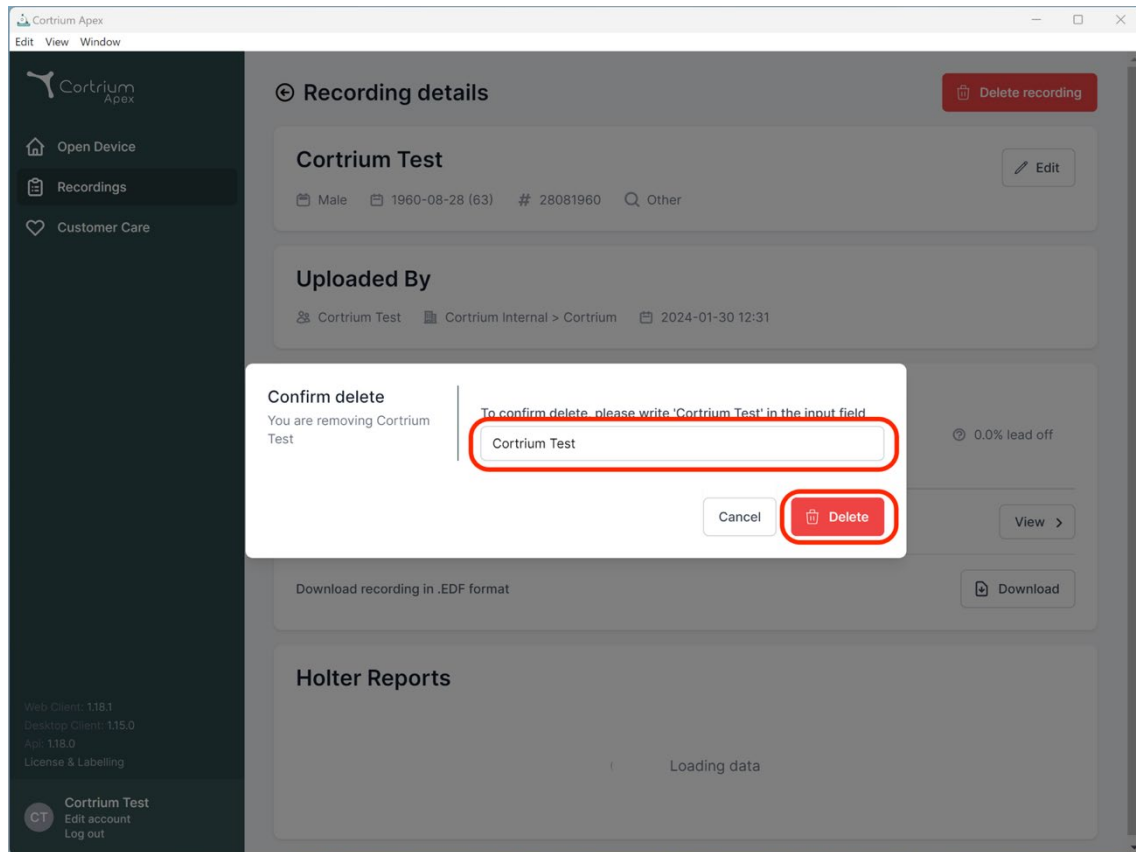
Features ordered: HRV AV-block Fast Track

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Confirm delete

You are removing Cortrium Test

To confirm delete, please write 'Cortrium Test' in the input field

Cancel Delete

0.0% lead off

View >

Download recording in .EDF format Download

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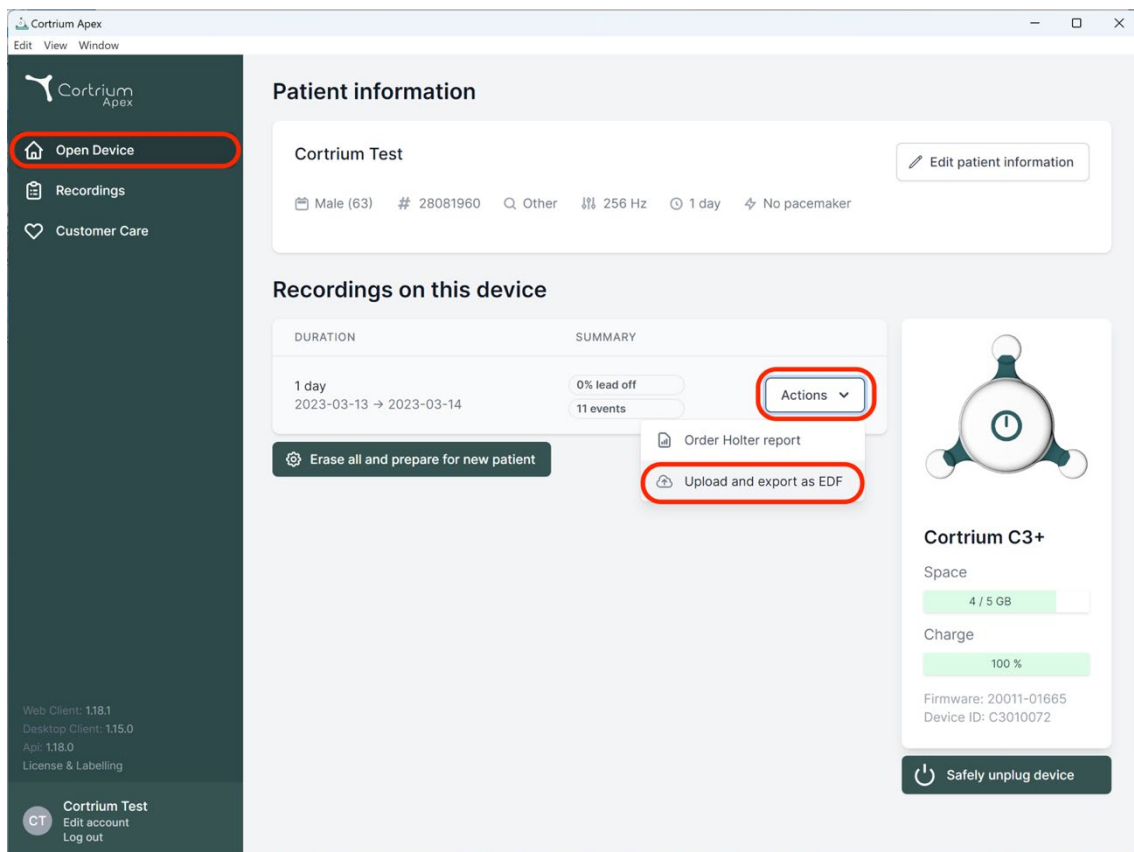
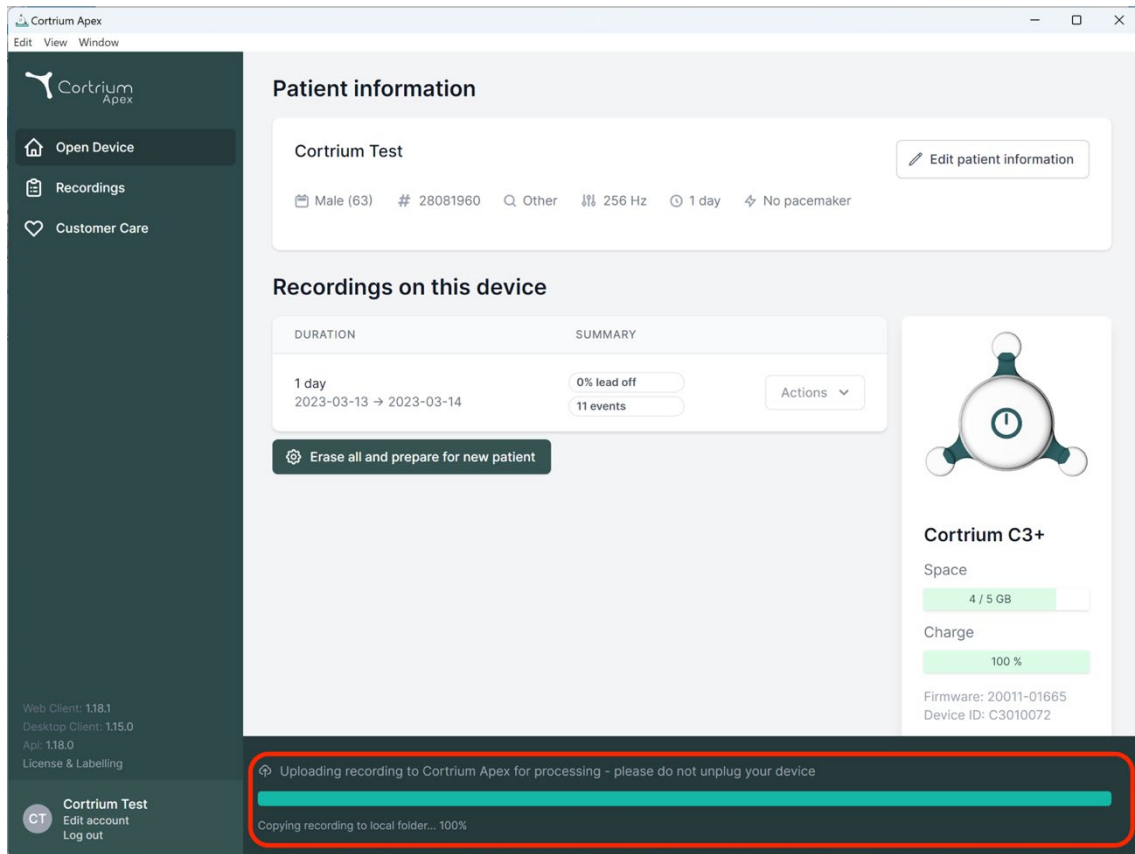
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

Uploading recording to Cortrium Apex for processing - please do not unplug your device

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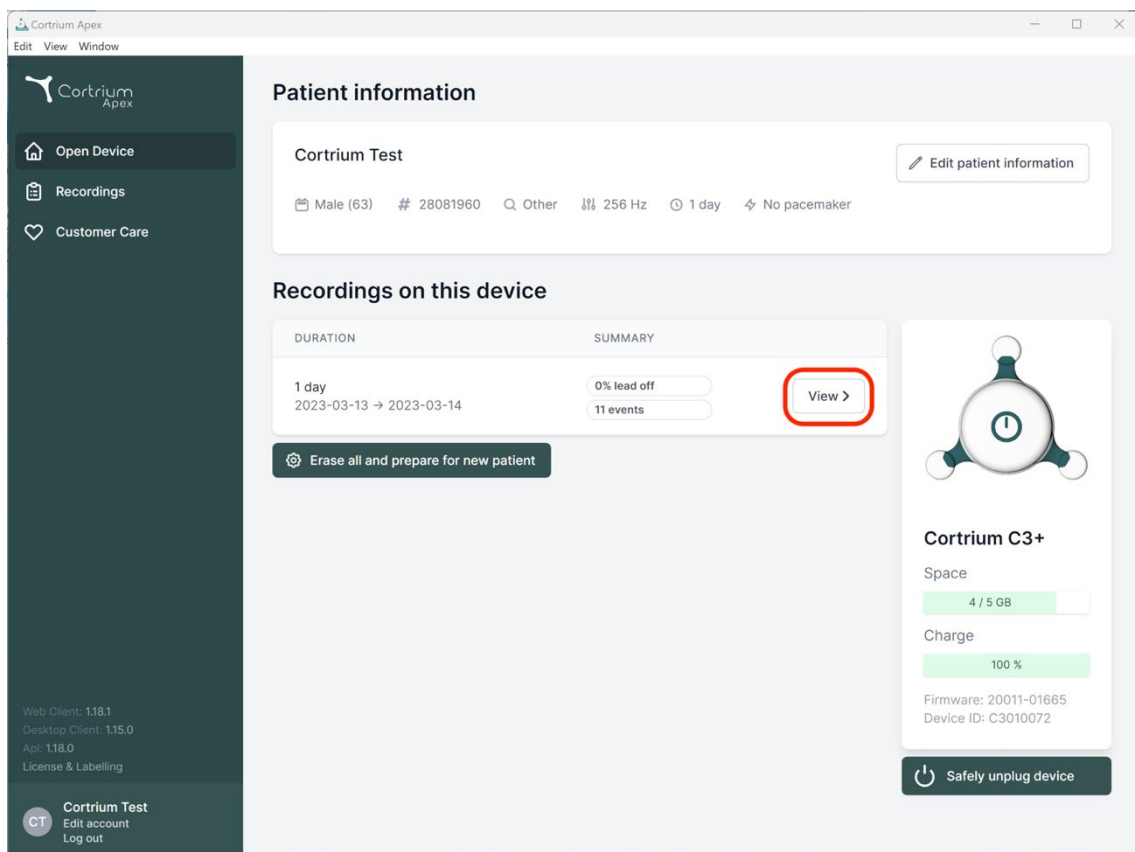
Male (63) # 28081960 Q Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY	Actions
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events	View >

[Erase all and prepare for new patient](#)

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
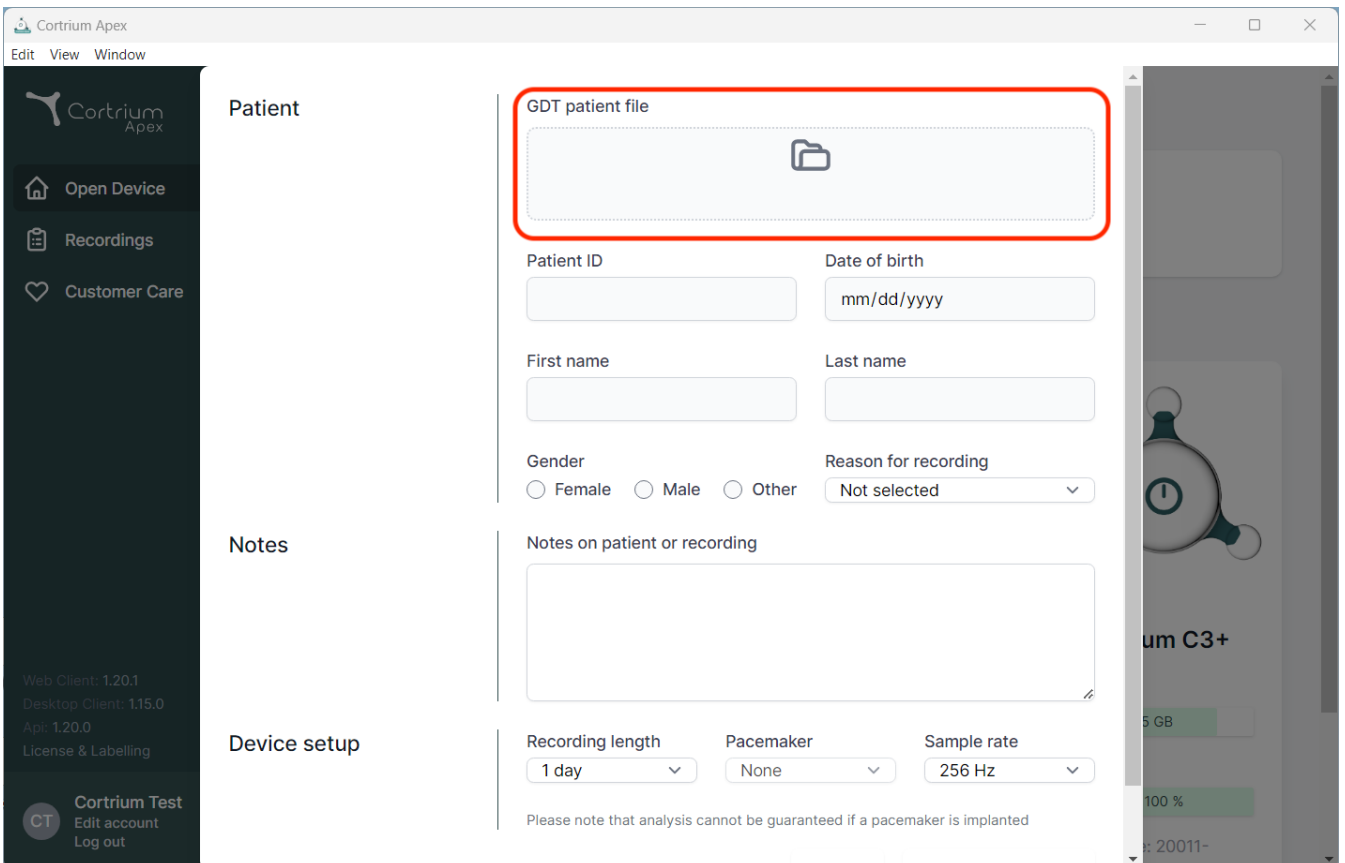
The screenshot displays the Cortrium Apex web application interface. The main content area is titled "Patient" and contains a form for entering patient information. A red rectangular box highlights the "GDT patient file" section, which includes a file upload icon. Below this, the form has several input fields: "Patient ID", "Date of birth" (with a placeholder "mm/dd/yyyy"), "First name", "Last name", "Gender" (with radio buttons for Female, Male, and Other), and "Reason for recording" (a dropdown menu currently set to "Not selected"). There is also a "Notes on patient or recording" text area. At the bottom, the "Device setup" section includes dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz"). A note below these settings states: "Please note that analysis cannot be guaranteed if a pacemaker is implanted". The left sidebar contains navigation options like "Open Device", "Recordings", and "Customer Care", along with version information and a "Cortrium Test" button.

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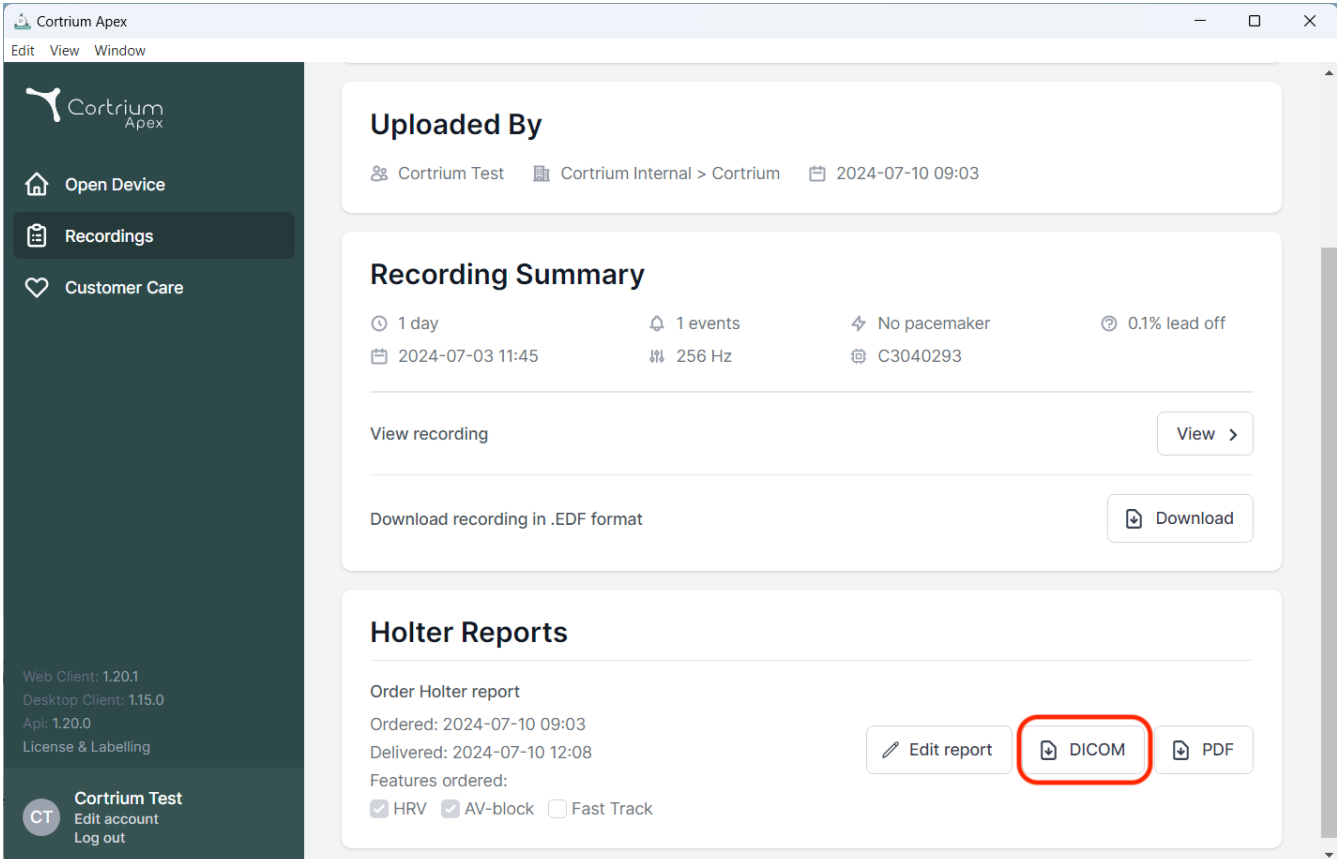
The screenshot displays the Cortrium Apex web application interface. On the left is a dark sidebar with navigation options: 'Open Device', 'Recordings', and 'Customer Care'. The main content area is divided into three sections:

- Uploaded By:** Shows 'Cortrium Test' and 'Cortrium Internal > Cortrium' with a timestamp of '2024-07-10 09:03'.
- Recording Summary:** Displays recording details: '1 day' (2024-07-03 11:45), '1 events', '256 Hz', 'No pacemaker', and '0.1% lead off' (C3040293). It includes buttons for 'View recording' and 'Download recording in .EDF format'.
- Holter Reports:** Shows 'Order Holter report' with dates 'Ordered: 2024-07-10 09:03' and 'Delivered: 2024-07-10 12:08'. It lists 'Features ordered: HRV, AV-block, Fast Track'. Three buttons are visible: 'Edit report', 'DICOM' (highlighted with a red circle), and 'PDF'.

At the bottom left of the sidebar, there is a user profile for 'Cortrium Test' with options to 'Edit account' and 'Log out'.

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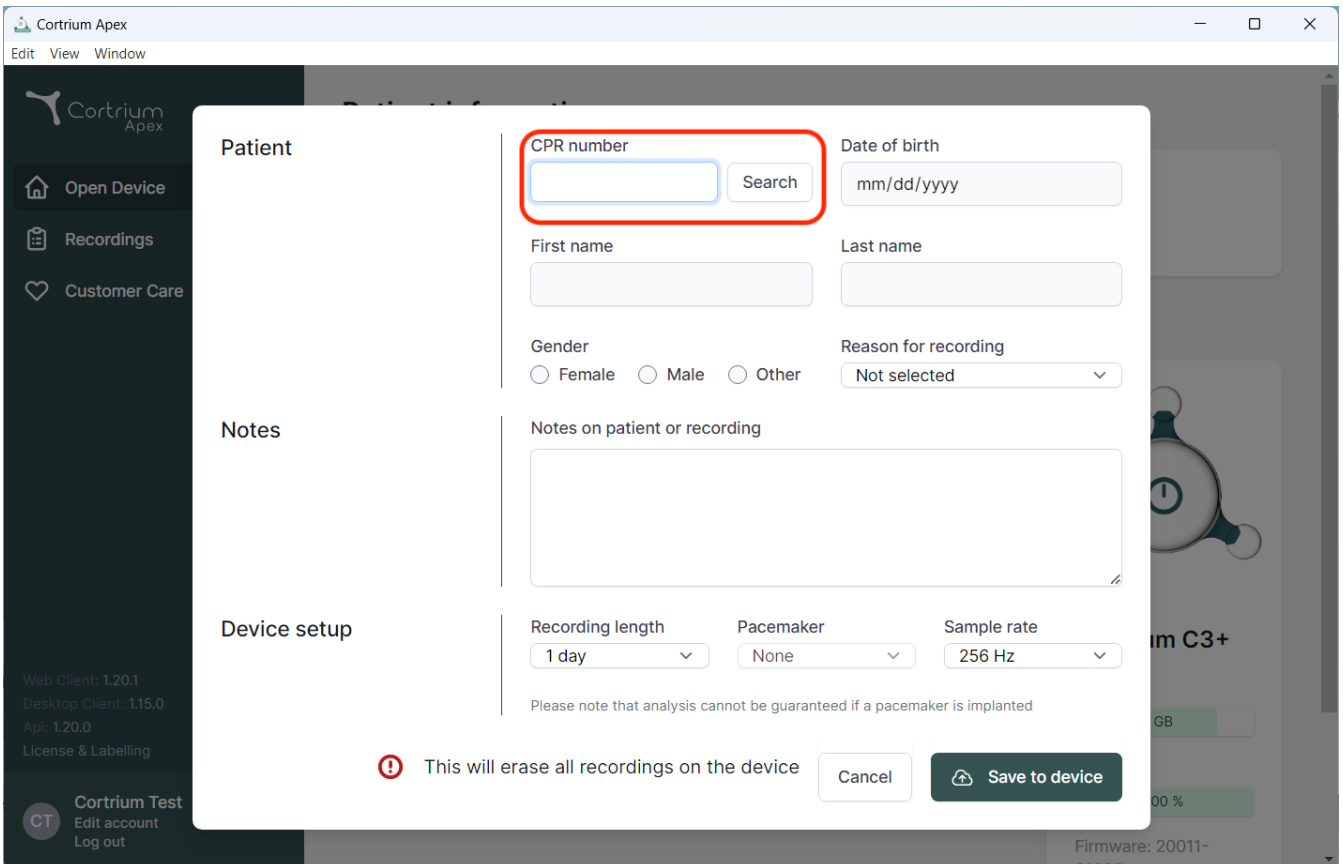The screenshot shows the Cortrium Apex web application interface. A modal window titled "Patient" is open, displaying a search form. The "CPR number" field is highlighted with a red box. The form includes the following fields and options:

- CPR number:** A text input field with a "Search" button next to it.
- Date of birth:** A text input field with a placeholder "mm/dd/yyyy".
- First name:** A text input field.
- Last name:** A text input field.
- Gender:** Radio buttons for "Female", "Male", and "Other".
- Reason for recording:** A dropdown menu with "Not selected" as the current option.
- Notes:** A text area labeled "Notes on patient or recording".
- Device setup:** Three dropdown menus for "Recording length" (set to "1 day"), "Pacemaker" (set to "None"), and "Sample rate" (set to "256 Hz").

Below the form, there is a warning message: "This will erase all recordings on the device" with a red warning icon. At the bottom right of the modal, there are two buttons: "Cancel" and "Save to device".

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