Cortrium Apex 1.22

Lietošanas instrukcija

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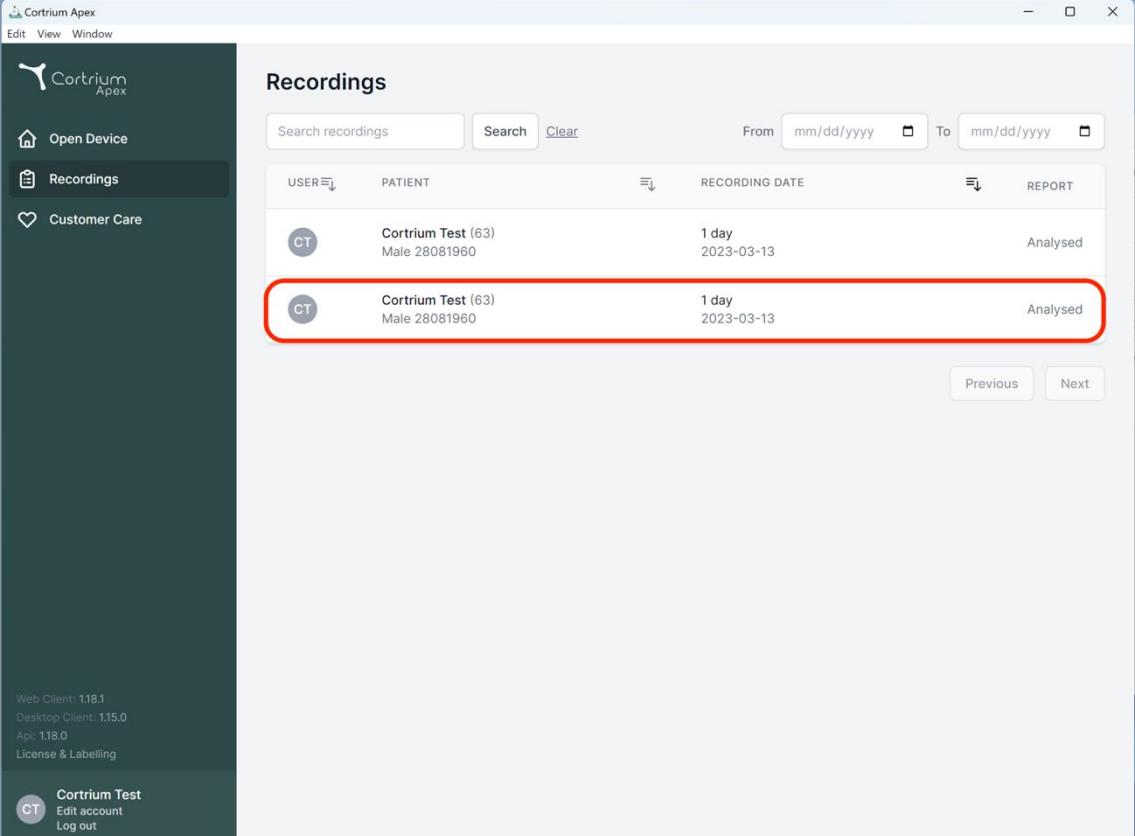
The screenshot shows the Cortrium Apex software interface. On the left is a dark sidebar with the Cortrium logo, 'Open Device' button, 'Recordings' button (which is highlighted in blue), and 'Customer Care' button. At the bottom of the sidebar are version numbers: 'Web Client: 1.18.1', 'Desktop Client: 1.15.0', 'ApI: 118.0', and 'License & Labelling'. Below that is a user profile for 'Cortrium Test (63)' with options to 'Edit account' or 'Log out'. The main window title is 'Recordings'. It features a search bar with 'Search recordings', 'Search' button, and date filters 'From' and 'To'. Below the search area is a table with columns: 'USER', 'PATIENT', 'RECORDING DATE', and 'REPORT'. There are two entries in the table:

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At the bottom right of the main window are 'Previous' and 'Next' buttons.

Cortrium Apex

Edit View Window

 Cortrium Apex

[Open Device](#)

[Recordings](#) Selected

[Customer Care](#)

Web Client: 1.18.1
Desktop Client: 1.15.0
API: 1.18.0
License & Labelling

 Cortrium Test
[Edit account](#) [Log out](#)

Recording details

Cortrium Test

[Edit](#) [Delete recording](#)

Male | 1960-08-28 (63) | # 28081960 | Other

Uploaded By

Cortrium Test | Cortrium Internal > Cortrium | 2024-01-30 12:31

Recording Summary

1 day | 11 events | No pacemaker | 0.0% lead off

2023-03-13 11:54 | 256 Hz | C3100175

[View recording](#) [View >](#)

Download recording in .EDF format [Download](#)

Holter Reports

[Order Holter report](#)

Ordered: 2024-02-16 07:36
Delivered: 2024-02-16 13:00
Features ordered:
 HRV AV-block Fast Track

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
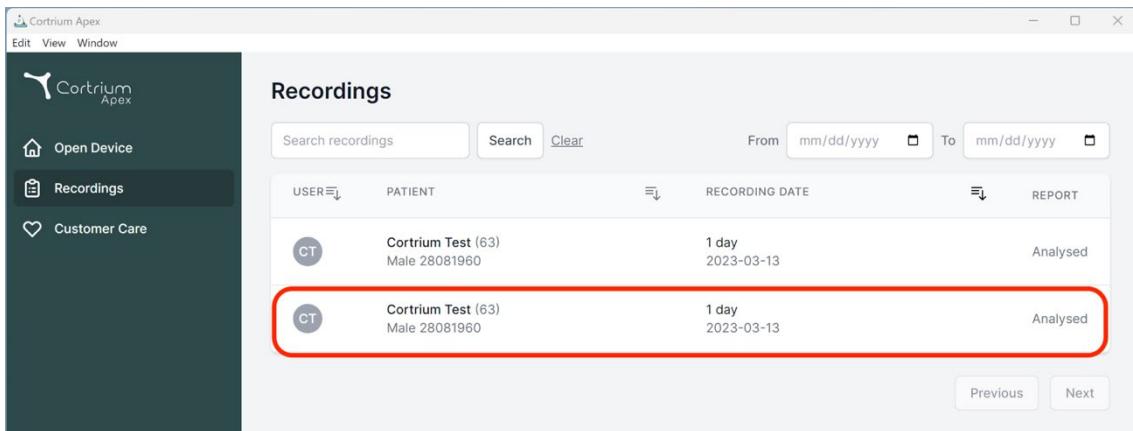
The screenshot shows the Cortrium Apex software interface. On the left is a dark sidebar with icons for 'Open Device', 'Recordings' (which is selected), and 'Customer Care'. The main area is titled 'Recordings' and contains a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. There are two entries:

USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

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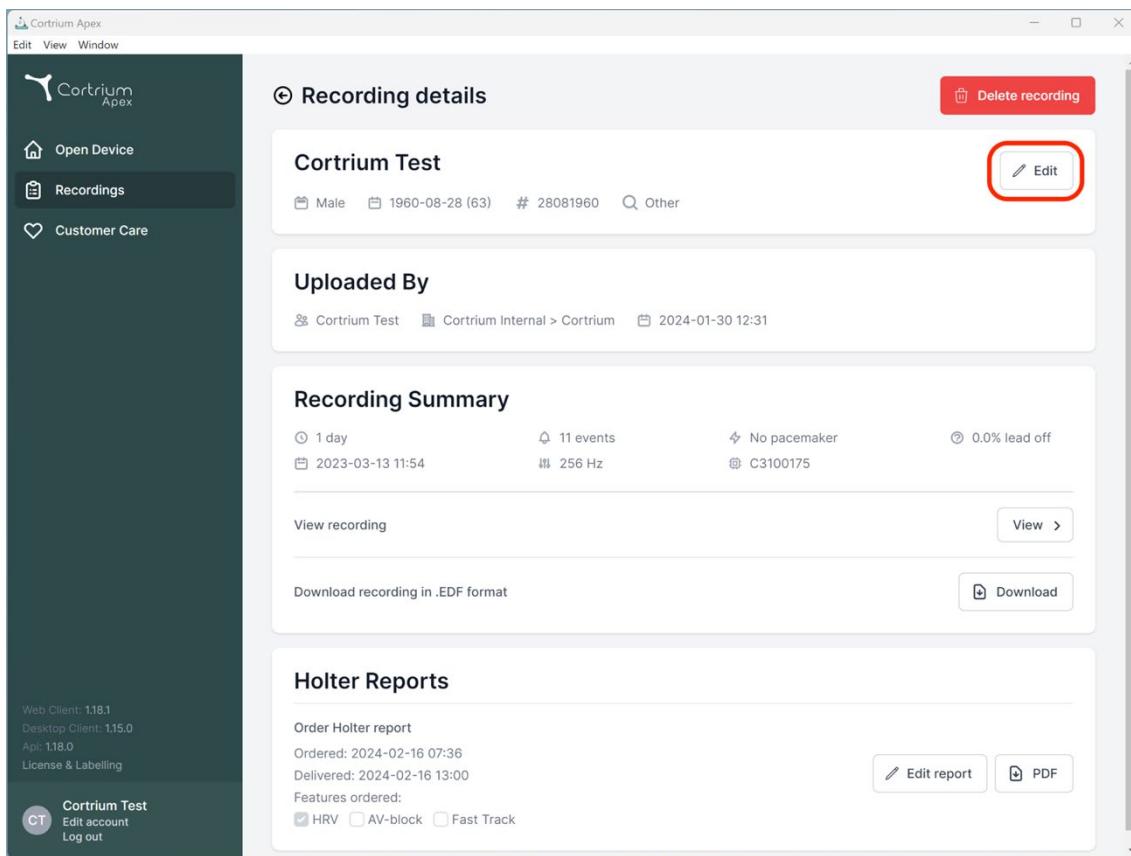The screenshot shows the 'Recording details' page for a recording named 'Cortrium Test'. The page includes sections for 'Recording details', 'Uploaded By', 'Recording Summary', and 'Holter Reports'. The 'Recording details' section shows the patient information: Male, born 1960-08-28 (63), ID 28081960, Other. The 'Edit' button is highlighted with a red box. The 'Recording Summary' section provides a summary of the recording: 1 day, 11 events, 256 Hz, No pacemaker, 0.0% lead off. The 'Holter Reports' section shows ordering details: Ordered: 2024-02-16 07:36, Delivered: 2024-02-16 13:00, Features ordered: HRV, AV-block, Fast Track. Buttons for 'Edit report' and 'PDF' are also present.

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Cortrium Apex

Edit View Window

Open Device Recordings Customer Care

Recording details

Cortrium Test

Male 1960-08-28 (63) # 28081960 Other

Patient ID: 28081960 Date of birth: 08/28/1960

First name: Cortrium Last name: Test

Gender: Male Reason for recording: Other

Notes on patient or recording:

Cancel Save

Order Holter report

Ordered: 2024-02-16 07:36 Delivered: 2024-02-16 13:00

Features ordered:

HRV AV-block Fast Track

View > Download Edit report PDF

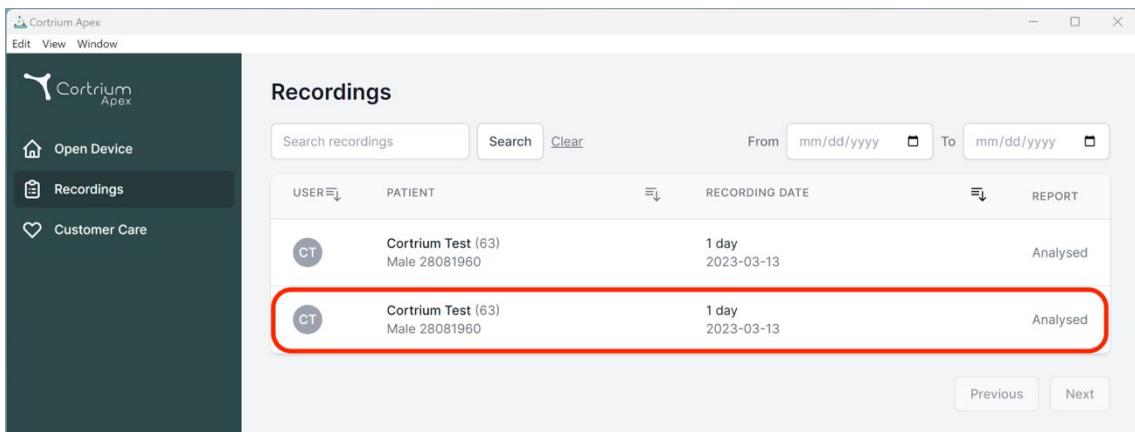
0.0% lead off

Save button highlighted with a red circle.

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
USER	PATIENT	RECORDING DATE	REPORT
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed
CT	Cortrium Test (63) Male 28081960	1 day 2023-03-13	Analysed

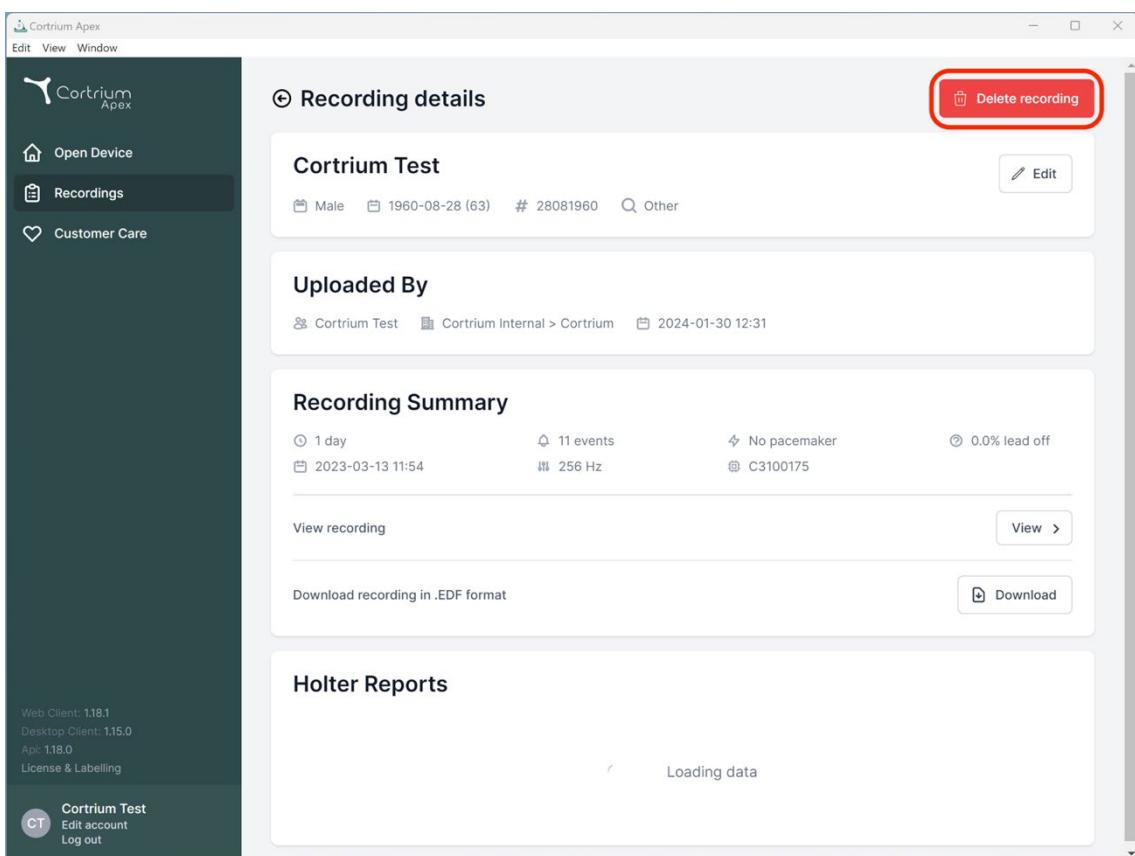Recording details

Cortrium Test

Male 1960-08-28 (63) # 28081960 Other

Uploaded By

Cortrium Test Cortrium Internal > Cortrium 2024-01-30 12:31

Recording Summary

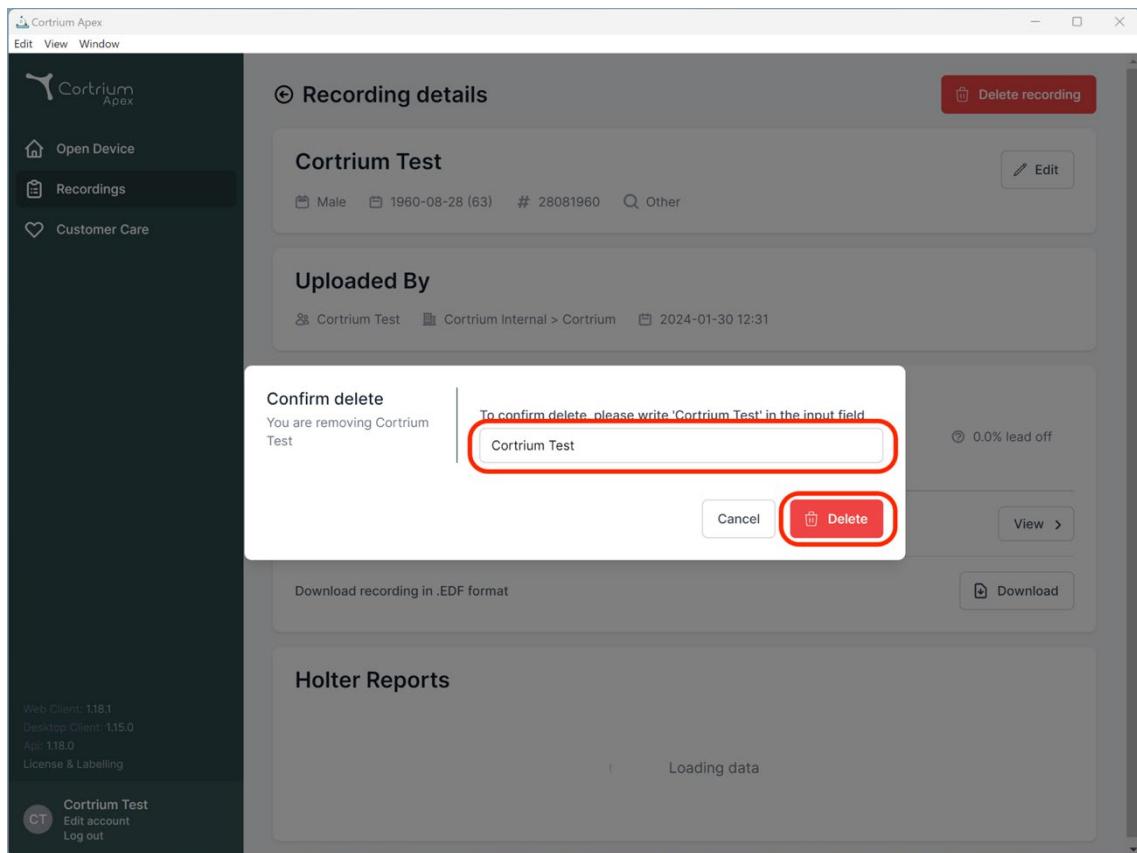
- 1 day
- 11 events
- No pacemaker
- 0.0% lead off
- 2023-03-13 11:54
- 256 Hz
- C3100175

[View recording](#) [View >](#)

[Download recording in .EDF format](#) [Download](#)

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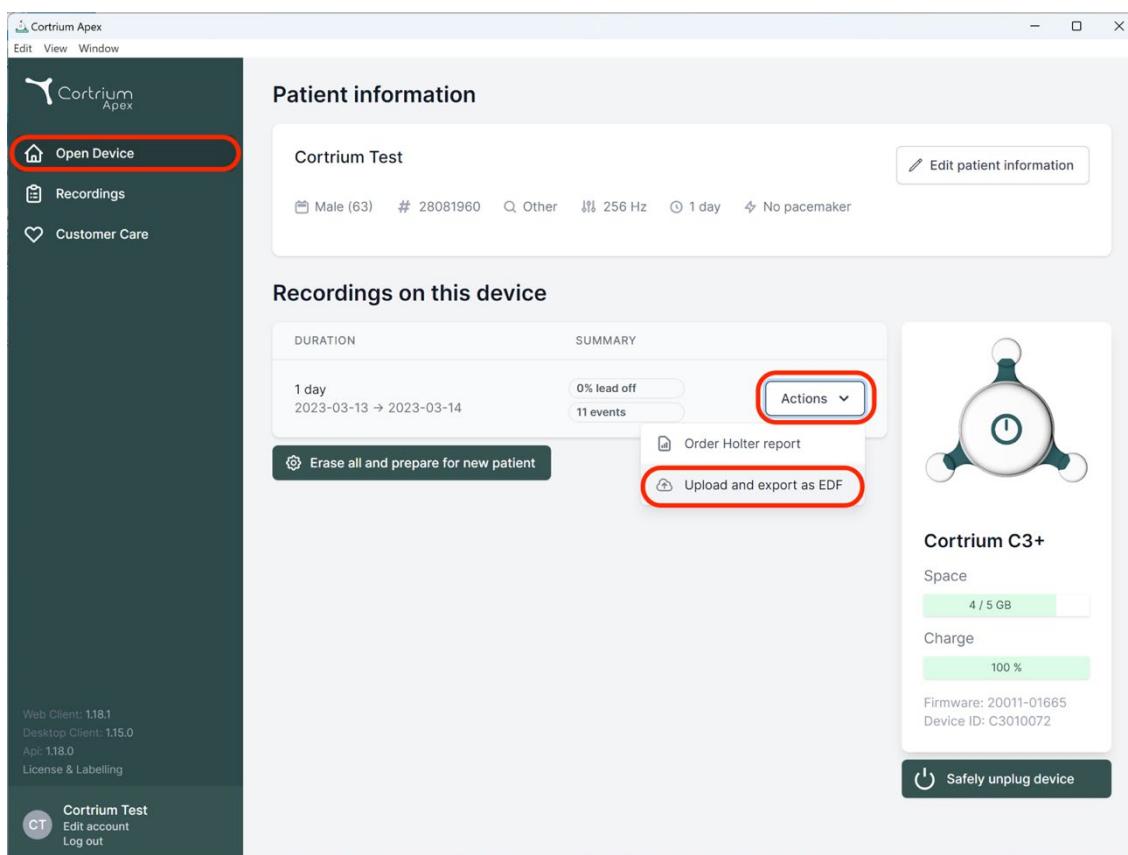
Male (63) # 28081960 Q, Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

Erase all and prepare for new patient

Cortrium C3+

Space: 4 / 5 GB
Charge: 100 %
Firmware: 20011-01665
Device ID: C3010072

Uploading recording to Cortrium Apex for processing - please do not unplug your device

Copying recording to local folder... 100%

Cortrium Apex

Patient information

Cortrium Test

Male (63) # 28081960 Q, Other 256 Hz 1 day No pacemaker

Recordings on this device

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events

Erase all and prepare for new patient

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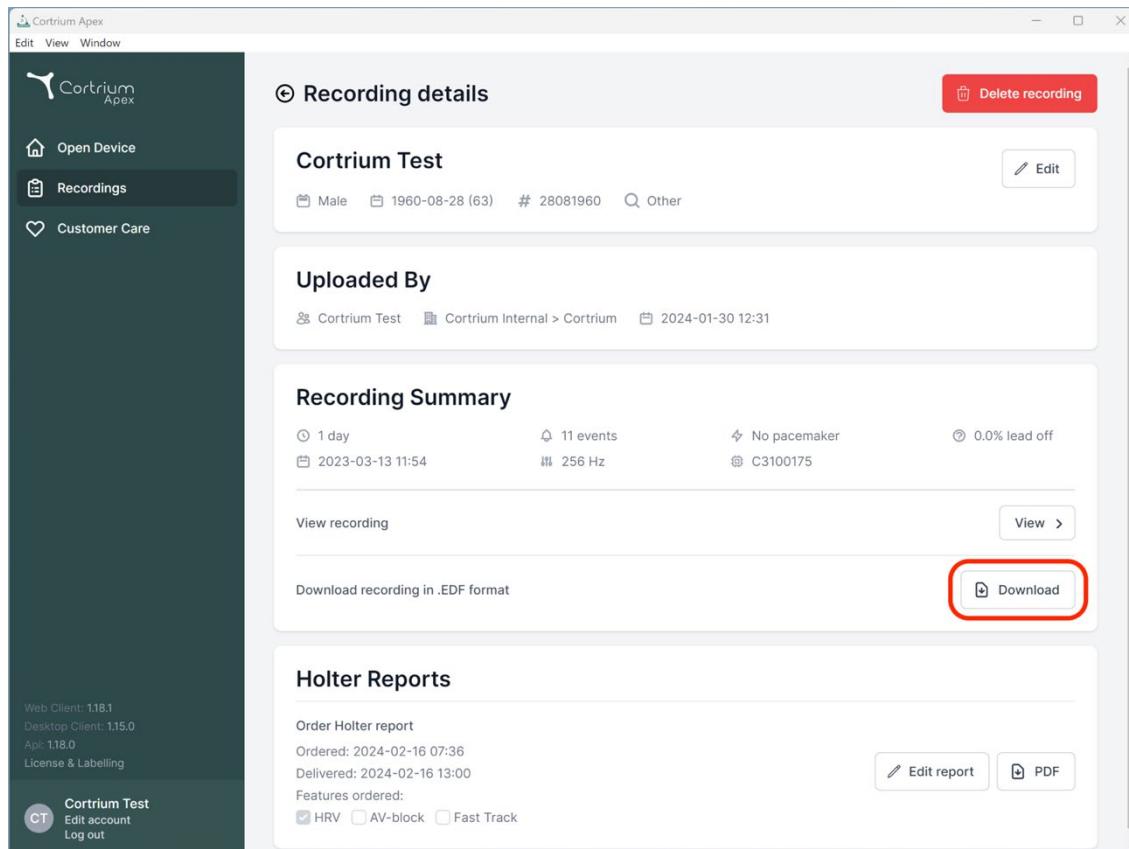
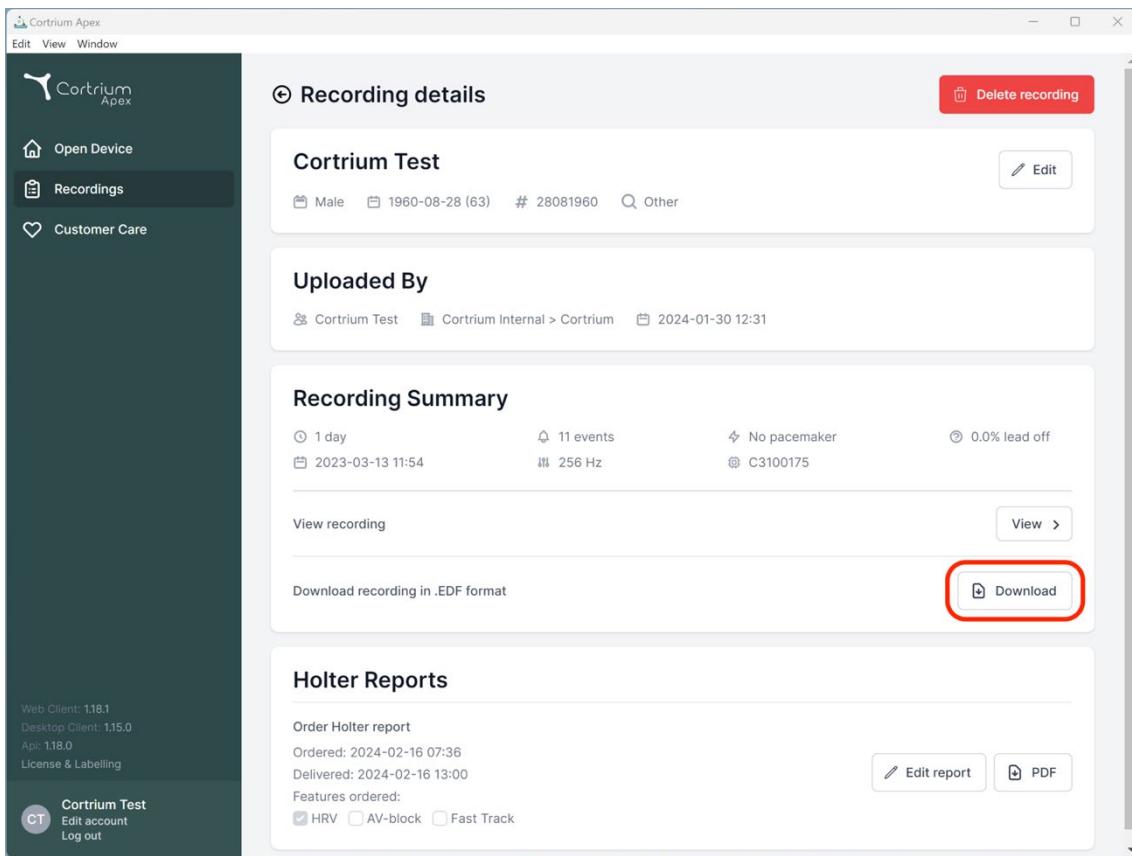
The screenshot shows the Cortrium Apex software interface. On the left is a dark sidebar with the Cortrium logo, navigation links (Open Device, Recordings, Customer Care), and user information (Web Client: 1.18.1, Desktop Client: 1.15.0, API: 1.18.0, License & Labelling). The main area has a light background. At the top, it says "Recording details" with a "Delete recording" button. Below that is a box for "Cortrium Test" with fields for gender (Male), date (1960-08-28 (63)), ID (28081960), and category (Other). There's an "Edit" button. Under "Uploaded By", it shows "Cortrium Test" uploaded via "Cortrium Internal > Cortrium" on 2024-01-30 12:31. A "Recording Summary" section follows, showing 1 day, 11 events, 256 Hz, no pacemaker, and 0.0% lead off. It includes a "View recording" link and a "Download recording in .EDF format" button, which is circled in red. The "Holter Reports" section shows an order for a Holter report, delivered on 2024-02-16 at 07:36, with features HRV, AV-block, and Fast Track selected. It includes "Edit report" and "PDF" buttons.

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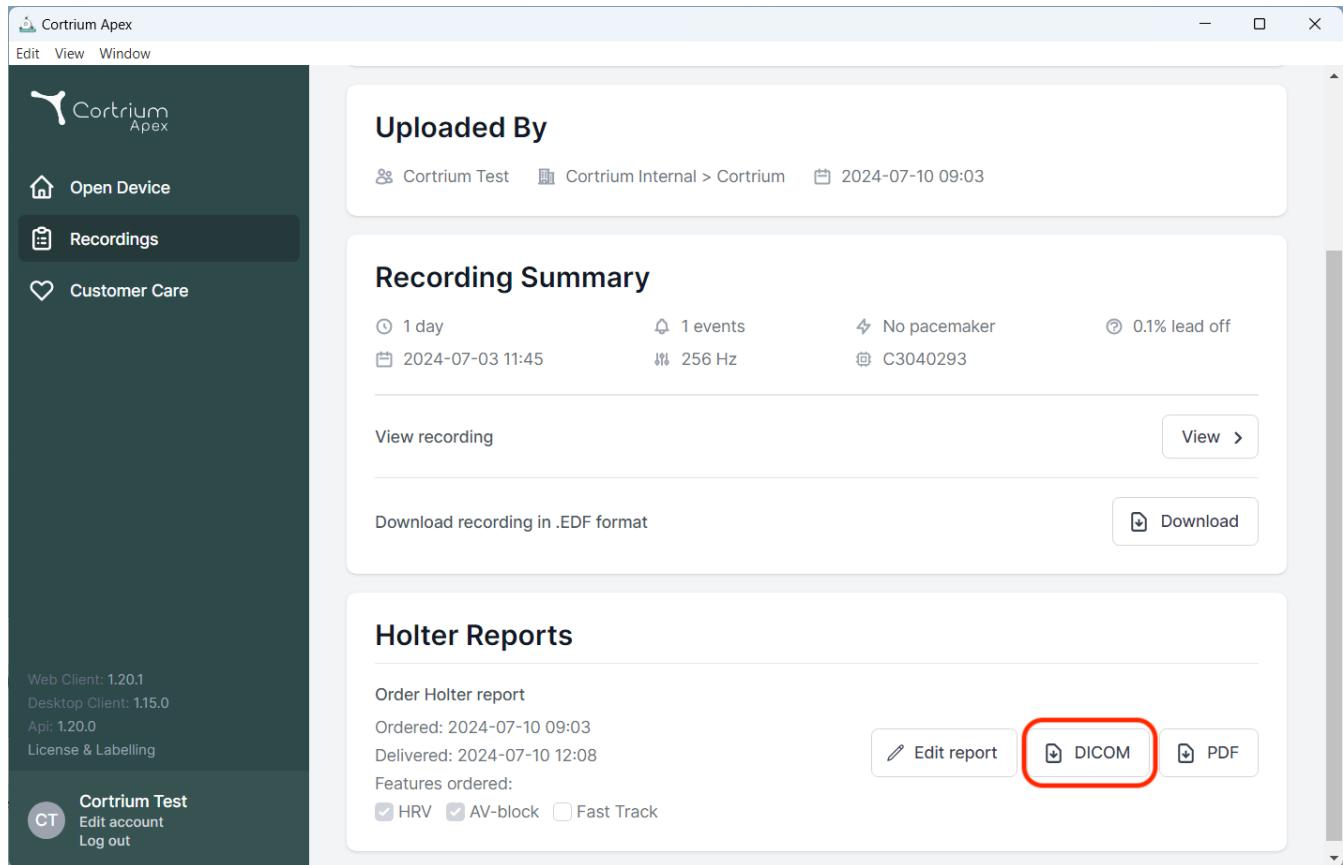
The screenshot shows the Cortrium Apex software interface. On the left, there's a sidebar with icons for Open Device, Recordings (which is selected), and Customer Care. Below that, system information is listed: Web Client: 1.20.1, Desktop Client: 1.15.0, API: 1.20.0, and License & Labelling. At the bottom of the sidebar is a user profile with initials 'CT' and options to Edit account and Log out.

The main area has a header 'Uploaded By' with details: Cortrium Test, Cortrium Internal > Cortrium, 2024-07-10 09:03.

Recording Summary section shows:

- 1 day duration
- 1 events recorded
- No pacemaker detected
- 0.1% lead off
- Date: 2024-07-03 11:45
- Sampling rate: 256 Hz

Buttons: View recording (with a 'View >' link) and Download recording in .EDF format (with a 'Download' button).

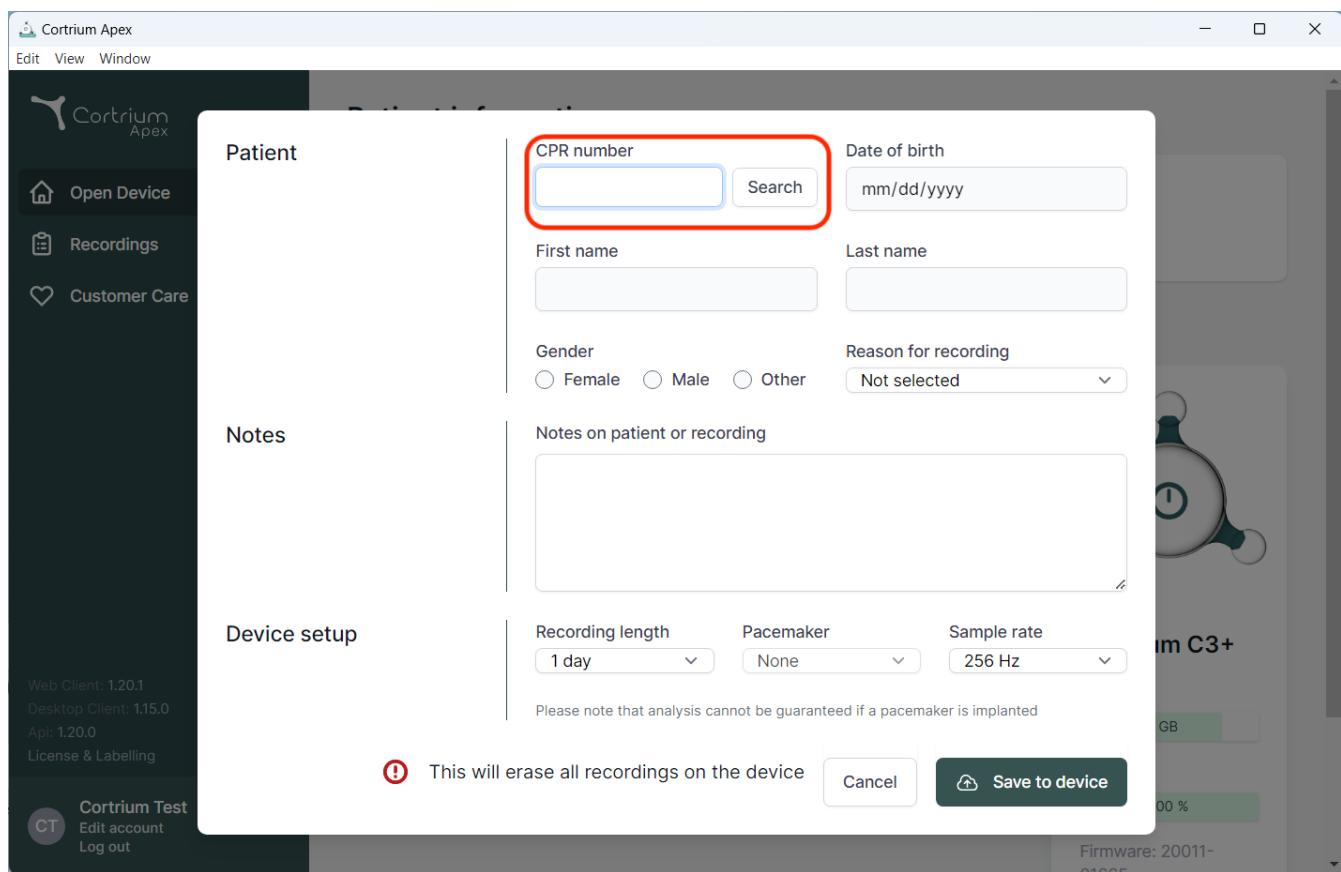
Holter Reports section shows:

- Order Holter report
- Ordered: 2024-07-10 09:03
- Delivered: 2024-07-10 12:08
- Features ordered:
 - HRV
 - AV-block
 - Fast Track

Buttons: Edit report, DICOM (which is highlighted with a red box), and PDF.

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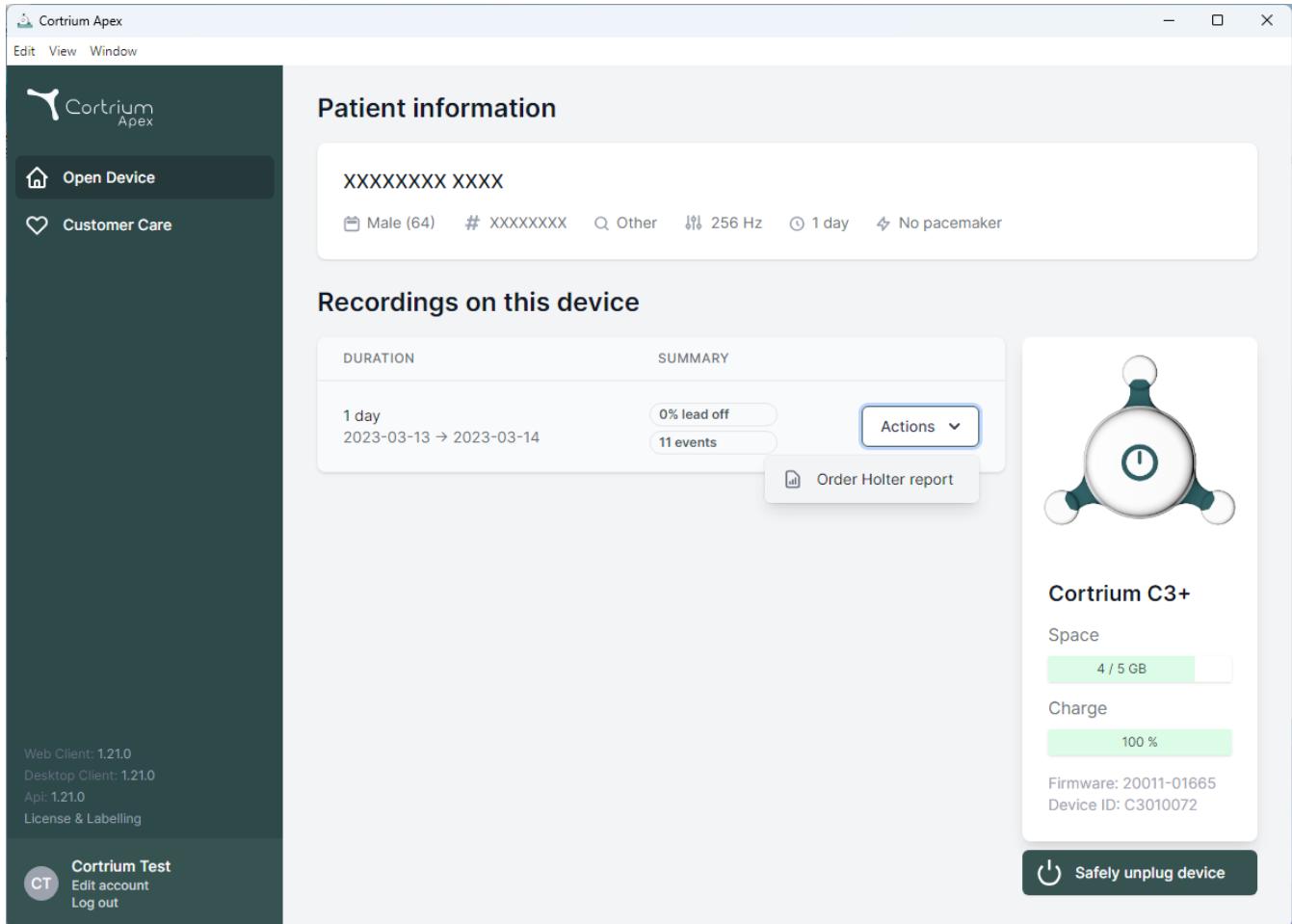
The screenshot shows the Cortrium Apex software interface. On the left is a dark sidebar with the Cortrium logo, navigation links like 'Open Device' and 'Customer Care', and system status: 'Web Client: 1.21.0', 'Desktop Client: 1.21.0', 'Api: 1.21.0', 'License & Labelling', and a user account section for 'Cortrium Test'.

The main content area has a title 'Patient information' with placeholder text 'XXXXXXXX XXXX'. Below it are demographic details: 'Male (64)', '# XXXXXXXX', 'Other', '256 Hz', '1 day', and 'No pacemaker'.


A section titled 'Recordings on this device' displays a summary table:

DURATION	SUMMARY
1 day 2023-03-13 → 2023-03-14	0% lead off 11 events Actions ▾ Order Holter report

To the right of the table is a circular icon representing the device, labeled 'Cortrium C3+'. It shows 'Space' usage at '4 / 5 GB' and 'Charge' at '100 %'. Below the device info is the text 'Firmware: 20011-01665' and 'Device ID: C3010072'. At the bottom right is a button with a plug icon labeled 'Safely unplug device'.

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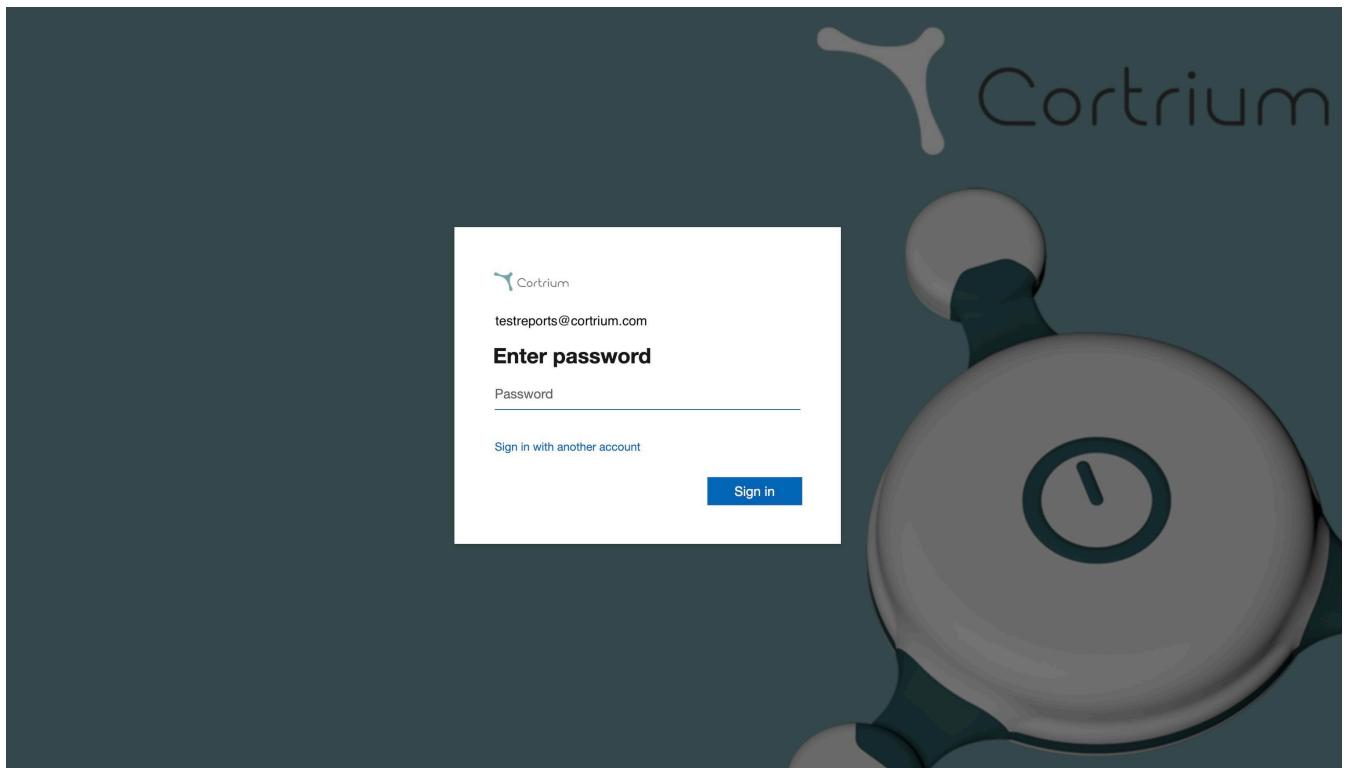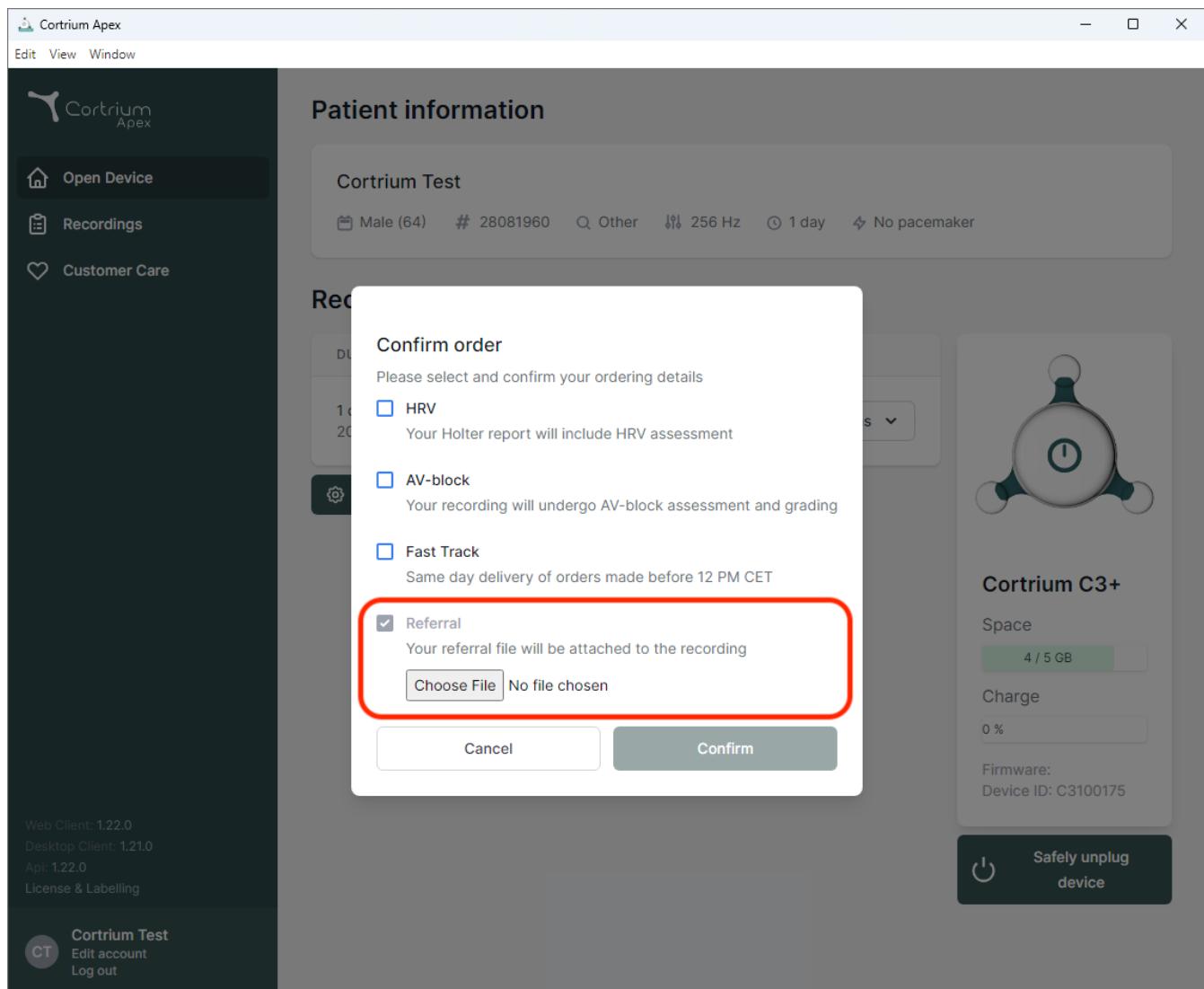
Referral

Recording Summary

32 minutes 0 events No pacemaker 0.0% lead off
2020-01-16 11:31 256 Hz C3020195

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
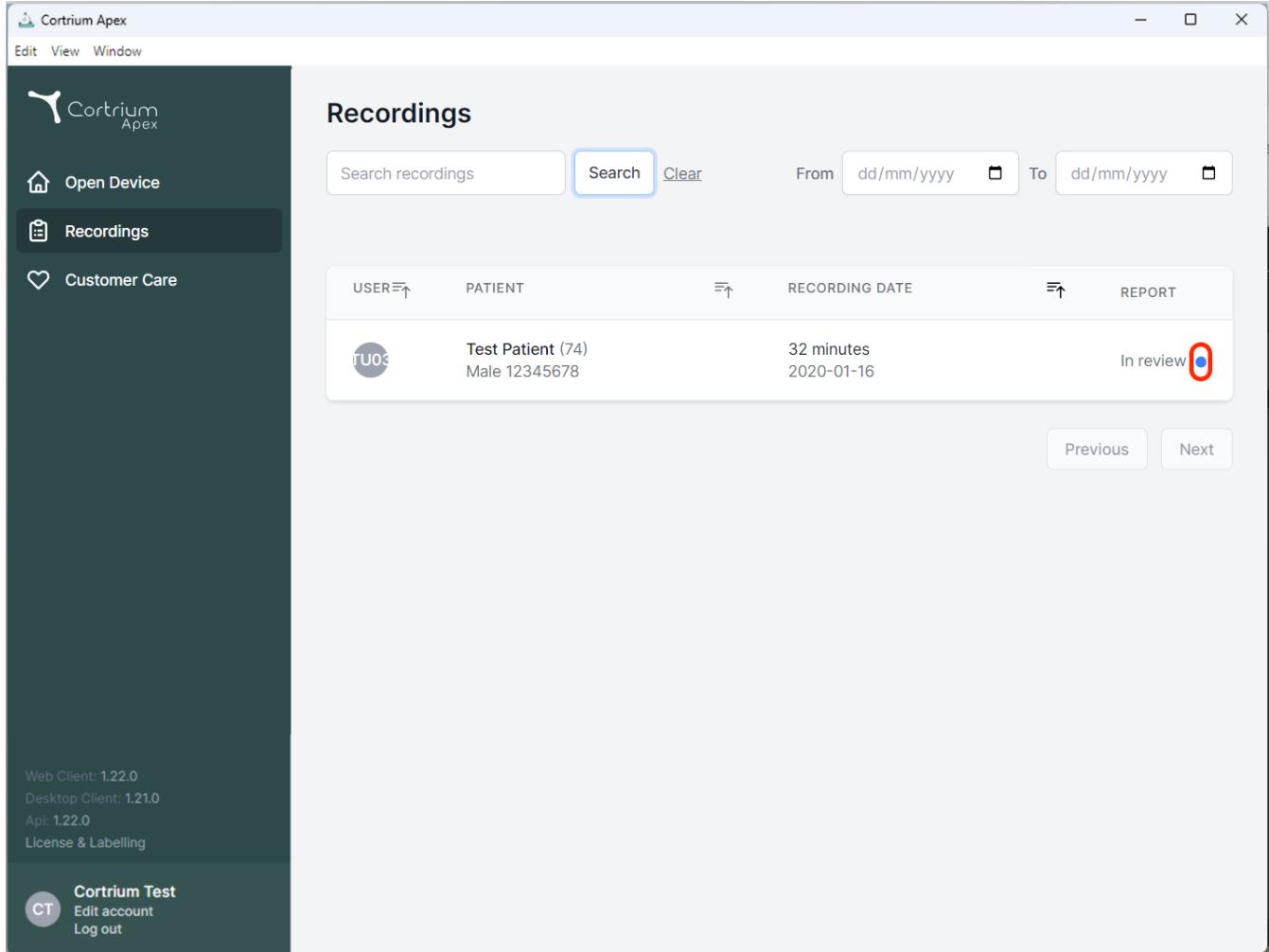
The screenshot shows the Cortrium Apex software interface. The left sidebar has a dark teal background with the Cortrium logo at the top, followed by menu items: "Edit", "View", and "Window". Below these are three main buttons: "Open Device" (with a house icon), "Recordings" (with a clipboard icon, which is highlighted in a darker shade), and "Customer Care" (with a heart icon). At the bottom of the sidebar, there are links for "Web Client: 1.22.0", "Desktop Client: 1.21.0", "Api: 1.22.0", and "License & Labelling". A user profile icon with "CT" and "Cortrium Test" is at the bottom right of the sidebar. The main content area has a light gray header with the title "Recordings". It contains a search bar with "Search recordings" and a "Search" button, a date range selector from "From dd/mm/yyyy" to "To dd/mm/yyyy", and a "Clear" button. Below the header is a table with the following columns: "USER" (TU03), "PATIENT" (Test Patient (74), Male 12345678), "RECORDING DATE" (32 minutes, 2020-01-16), and "REPORT" (In review, with a red circle around the "0"). At the bottom of the table are "Previous" and "Next" buttons. The overall interface is clean and modern, using a combination of dark and light colors.

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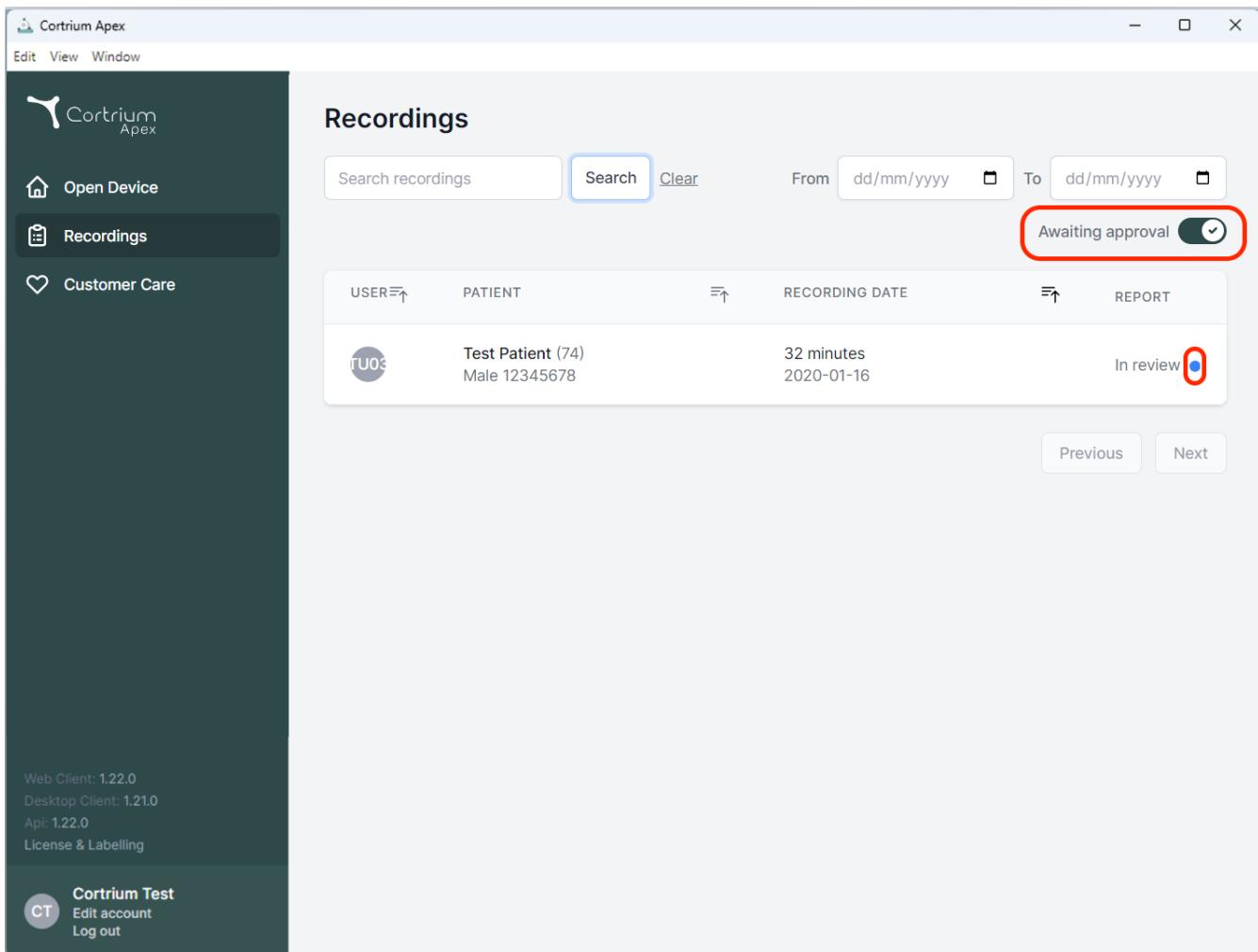
The screenshot shows the Cortrium Apex software interface. On the left is a dark sidebar with icons for Open Device, Recordings (which is selected), and Customer Care. At the bottom of the sidebar, it displays system information: Web Client: 1.22.0, Desktop Client: 1.21.0, API: 1.22.0, and License & Labelling. On the right is the main 'Recordings' screen. It features a search bar with 'Search recordings' and a 'Search' button, followed by date filters 'From dd/mm/yyyy' and 'To dd/mm/yyyy'. Below these is a toggle switch labeled 'Awaiting approval' with a checked state, also highlighted by a red box. The main area displays a table with columns: USER, PATIENT, RECORDING DATE, and REPORT. One row is visible for a user named 'TU03' who recorded a 'Test Patient (74)' male patient with ID 12345678. The recording duration was 32 minutes on 2020-01-16. The 'REPORT' column shows the status 'In review' with a small red circle containing a blue dot. At the bottom of the main screen are 'Previous' and 'Next' navigation buttons.

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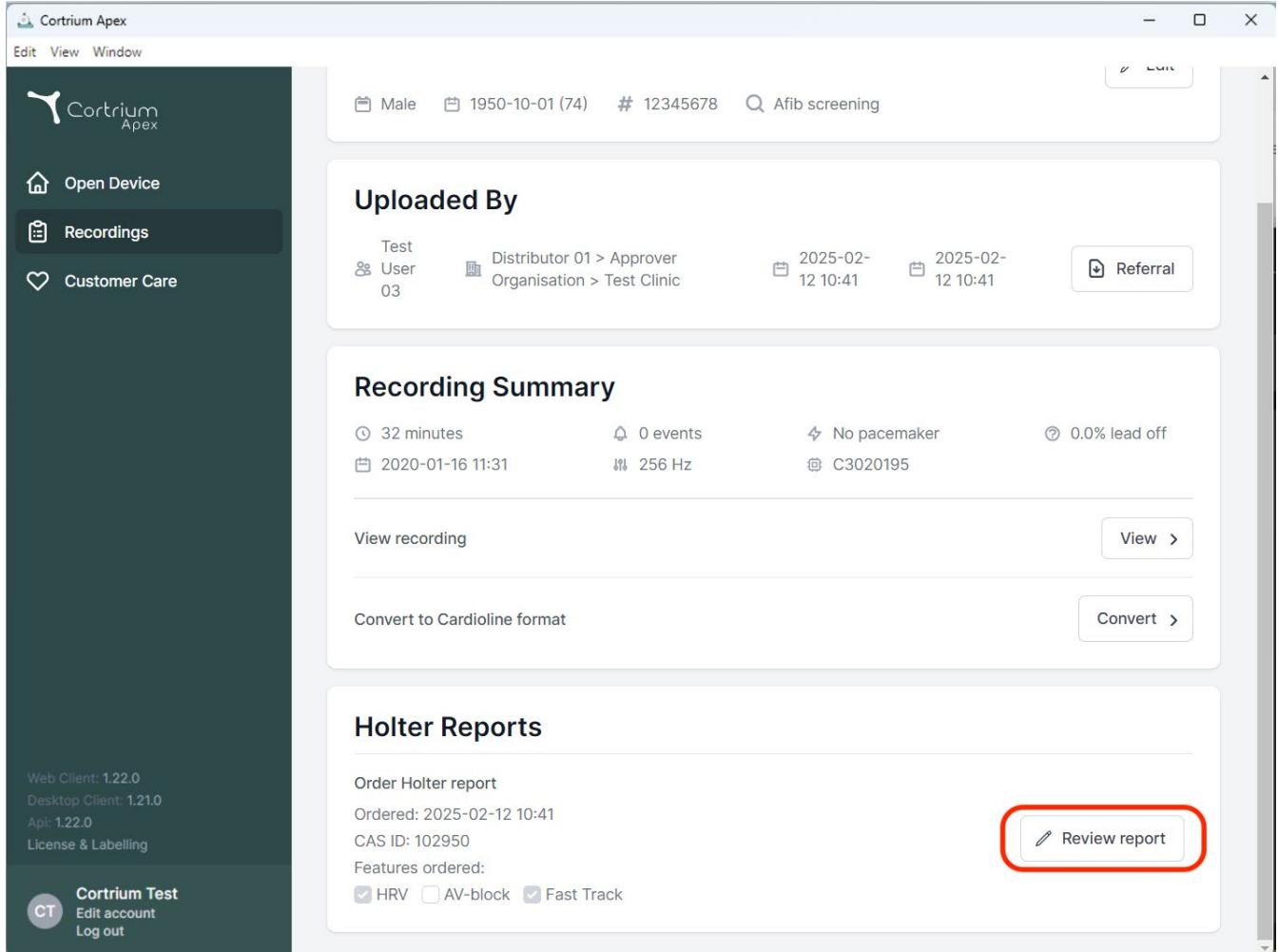The screenshot shows the Cortrium Apex software interface. On the left, there's a sidebar with navigation links: Open Device, Recordings (which is selected), and Customer Care. Below that, system information like Web Client version (1.22.0) and a user icon labeled 'CT'.

The main content area has a header with patient details: Male, Date of Birth: 1950-10-01 (74), Record ID: 12345678, and Search term: Afib screening. There's also a 'Edit' button.

Uploaded By:

Test User 03	Distributor 01 > Approver Organisation > Test Clinic	2025-02-12 10:41	2025-02-12 10:41
--------------	--	------------------	------------------

Recording Summary:

⌚ 32 minutes	🔔 0 events	⚡ No pacemaker	ⓘ 0.0% lead off
📅 2020-01-16 11:31	㎐ 256 Hz	⌚ C3020195	

Buttons for 'View recording' and 'Convert to Cardioline format'.

Holter Reports:

Order Holter report
Ordered: 2025-02-12 10:41
CAS ID: 102950
Features ordered:
 HRV AV-block Fast Track

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Cortrium Test
Edit account
Log out

Summary

The duration of the recording is 32 min. 100% of the signal was of sufficient quality for analysis.
The average heart rate was 65 bpm. The fastest heart rate was 83 bpm at 11:43 (Day 1).
The slowest heart rate was 55 bpm at 12:02 (Day 1).


Report has been approved by a cardiologist.
Name of cardiologist: Test User 02.
Notes: Some notes.

Approve report


Examination overview

Start	End	Duration
2020-01-16, 11:31:59	2020-01-16, 12:04:06	32 min

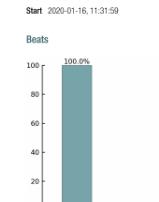Beats

Rhythms

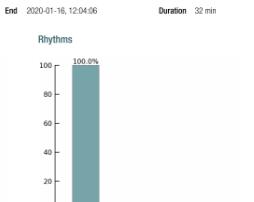Heart rate

Total beats	Total V beats	Total S beats
2081	0	0
Average HR	Couples	Couples
65 bpm	Runs	Runs
Slowest HR	Tachycardias	Tachycardias
55 bpm	Bigemines	Bigemines
Fastest HR	Trigemines	Trigemines
83 bpm	Idioventricular	AFIB/AFL

Ventricular

Supraventricular

02/2025

Cortrium ApS • Erik Husfeldts Vej 7 • DK-2630 Taastrup • Dānija
E-pasts: info@cortrium.com • Tīmekļa vietne: www.cortrium.com

Cortrium Apex UDI(01)05745000379033

CE
0123